

PushTracker E2 — Wear OS by Google set-up

NOTE: Wi-Fi is required for Wear OS by Google set-up. If doing a Wear OS by Google set-up on a PushTracker E2 that was preconfigured by Permobil, you must do a factory reset first.

1 PushTracker E2 & Wear OS by Google

1. Download *Wear OS by Google* on your Android or Apple phone



2. Read/accept Google terms of service & enable Bluetooth® sharing

3. Turn on watch, select language

4. Read/accept Mobvoi policy







5. Pair/connect PushTracker E2 to phone

6. Connect your watch to Wi-Fi

7. Choose/sign in/create Google account

8. Initialization

9. Wear OS tutorial

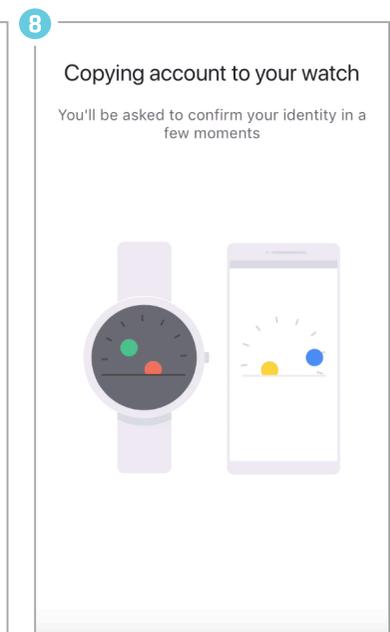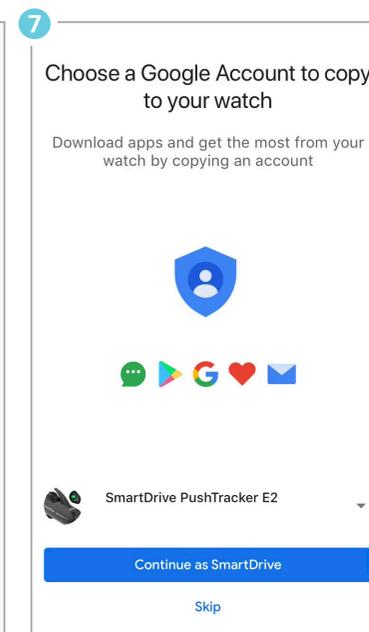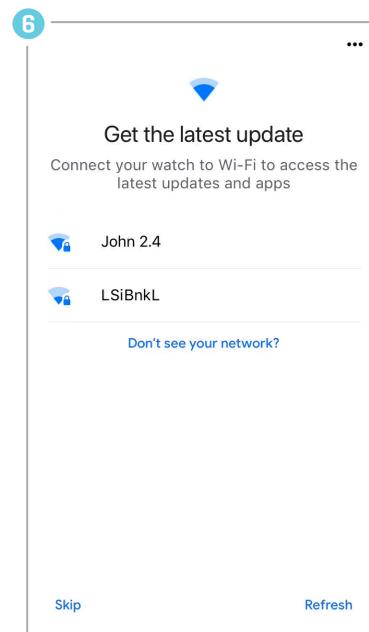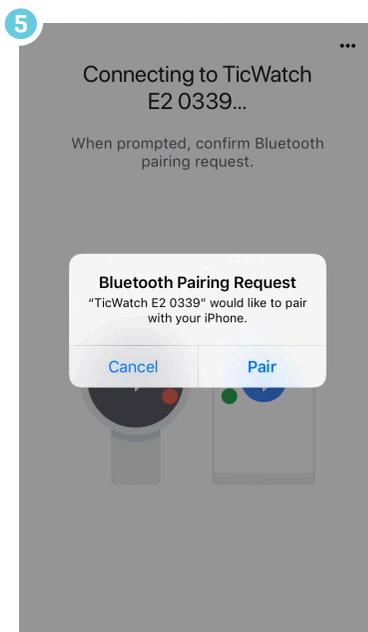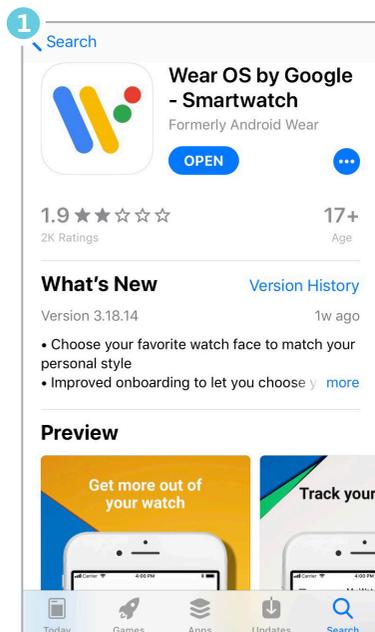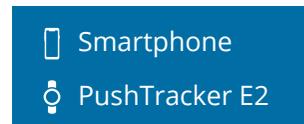

PushTracker E2 — Wear OS by Google set-up (cont.)

NOTE: Wi-Fi is required for Wear OS by Google set-up. If doing a Wear OS by Google set-up on a PushTracker E2 that was preconfigured by Permobil, you must do a factory reset first.

2 Download SmartDrive MX2+ app



1 Go to Google Play Store on PushTracker E2



2 Search for "SmartDrive MX2+"




3 Install & launch SmartDrive MX2+ app


3 Pair & go



1 With your SmartDrive MX2+ unit turned on, launch the SmartDrive MX2+ app and touch "TURN POWER ASSIST ON"

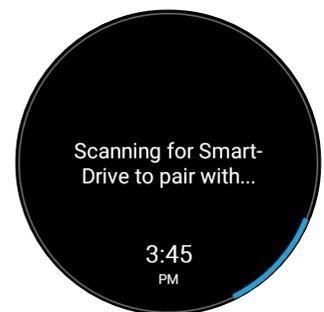

2 Pair with SmartDrive MX2+ by selecting the number listed on the screen. You can only pair when one SmartDrive MX2+ is present

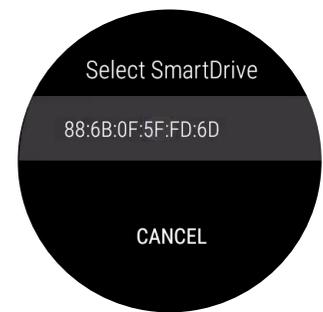

3 You are now paired & connected!

NOTE: You can also pair to your SmartDrive MX2+ through the settings menu within the app.

For more instructions check out the PushTracker E2 how-to videos at permobilus.com/howtosmartdrive.



*Wear OS by Google, iPhone, Android, Bluetooth and Google Play are trademarks or registered trademarks of their respective owners.