
ROHO®

AGILITY® UniTrack Rail

Operation Manual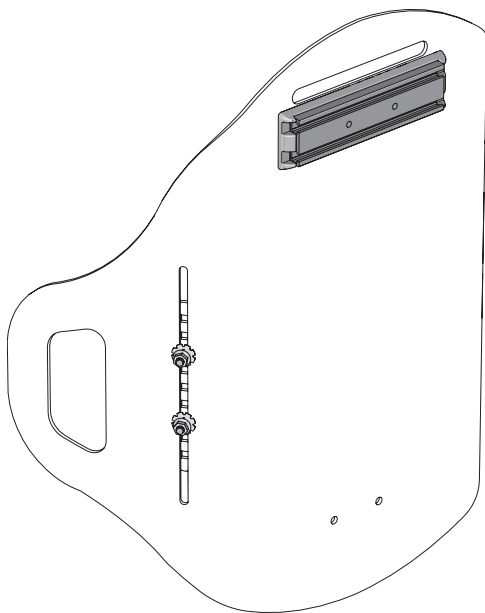

permobil

s h a p e f i t t i n g t e c h n o l o g y®

Contact Information

ROHO, Inc.
1501 South 74th St.
Belleville, IL 62223 U.S.A.
U.S.A.:
800-736-0925 • Fax 888-551-3449
Customer Support:
orders.roho@permobil.com
Outside the U.S.A.:
618-277-9150 • Fax 618-277-6518
intlorders.roho@permobil.com
permobil.com

**Warning**

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

**Caution**

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

Incident Reporting: In the event of a serious incident or injury, contact Customer Support.

Contents

Intended Use	2-3
Important Safety Information	
Parts Detail, Attachment, Removal	3
Troubleshooting	4
Cleaning and Disinfecting	
Storage, Transport, Disposal and Recycling	
Limited Warranty	
Español	5
Français	9
Deutsch	13
Italiano	17
Nederlands	21
Dansk	25
Svenska	29
Norsk	33
Polski	37
Português	41
Images	45

Symbols Glossary

Manufacturer

Date of manufacture

Authorized representative
in the European Community

Medical Device

Quantity

Authorized representative
in Switzerland

Batch code

Model number

Consult instructions
for use**Please Note**

Supplier: This manual must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Products may be covered by one or more U.S. and foreign patents and trademarks, including AGILITY® ROHO® and shape fitting technology®.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

ROHO, Inc. is part of Permobil.

© 2018, 2022 Permobil Inc.

Rev. 2022-10-24

Intended Use

Intended Use: The ROHO AGILITY UniTrack Rail (AGILITY UniTrack Rail) is an interface rail intended to attach the hardware for a Permobil head support to compatible ROHO AGILITY Back Supports: AGILITY Minimum Contour, AGILITY Mid Contour and AGILITY Max Contour.

Compatibility: AG-HR-UT14 fits only AGILITY 14xx widths. AG-HR-UT fits all other AGILITY widths.

A clinician who is experienced in seating and positioning should determine whether the product is appropriate for the individual's particular seating needs. The product's intended use statement alone should not be relied upon to make that determination.

The medical products featured in this document are intended to be only one part of an overall care regimen that includes all seating and mobility equipment and therapeutic interventions. A clinician who is experienced in seating and positioning should identify this care regimen after evaluating the individual's physical needs and overall medical condition.

A clinician should also evaluate for visual, reading, and cognitive impairment to determine the need for caregiver assistance or the need for other assistive technology, such as large-print instructions, to ensure proper product use.

Important Safety Information

Warnings:

- DO NOT perform any tasks without first reading and understanding all of the safety information and provided instructions. If you are unable to perform any tasks in this manual, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.
- Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- The product should be installed by the equipment provider or a clinician.
- As with any addition to a wheelchair, the product may affect the wheelchair's stability. Before using the product, consult with a clinician or equipment provider to determine whether the wheelchair should be modified.
- Check daily to make sure hardware is fully tightened. DO NOT use if any part cannot be tightened or is broken. Periodically, the product should be thoroughly inspected by a qualified technician.
- DO NOT expose the product to high heat, open flames or hot ashes. Testing or certification claims, including for flammability, may no longer apply to this device when it is combined with other products or materials. Check testing and certification claims for all products used in combination with this device.
- The product will acclimate to the surrounding temperature. Take appropriate precautions, especially when the product will contact unprotected skin.
- DO NOT modify the product or use non-compatible accessories or components, which may result in product damage or failure.
- Make sure the AGILITY UniTrack Rail is level. Use a leveling tool if needed.
- Follow the torque specifications recommended in these instructions. Unless otherwise noted, these instructions do not replace any other information provided with the AGILITY.
- The AGILITY UniTrack Rail must be correctly sized to the AGILITY back shell.

Important Safety Information, continued

Warning: Keep small hardware components out of the reach of children and any individuals who have a tendency to place inedible objects in their mouths. Swallowing or inhalation may lead to serious injury or death. IMMEDIATELY seek emergency medical assistance.

Motor Vehicle Transportation Warnings:

- Whenever possible, transfer out of the wheelchair installed with AGILITY and into a manufacturer-installed vehicle seat, and use the vehicle's crash-tested occupant restraint system.
- To determine whether motor vehicle third-party testing has been performed with a specific AGILITY-hardware and other accessory combination, refer to the operation manual provided with AGILITY, attachment hardware, and other accessory, or contact the equipment manufacturer.

Cautions:

- Protect the AGILITY from blunt force shocks that may cause damage and/or breakage.
- The components of an AGILITY UniTrack Rail are not intended to be used separately or in conjunction with non-compatible wheelchair back supports or accessories.

Parts Detail


Package Contents (Specifications): AGILITY UniTrack Rail (aluminum, polycarbonate-ABS alloy); two button-head screws (M6 x 14mm); two stainless steel washers (M6); instructions
Expected life: 60 months

To order replacement parts, contact the equipment provider or Customer Support.

Attachment

For attachment, refer to the images on page 45 in this manual.

1. Remove the AGILITY Cover from the back shell.
2. Place the two washers on the two screws. Insert the screws through the openings in the back shell and the AGILITY UniTrack Rail.
3. Make sure the AGILITY UniTrack Rail is level. Use a leveling tool if needed.
4. Use a 4mm allen wrench to securely tighten the screws.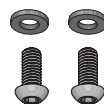

Washers and screws

Note: If a torque wrench is available, tighten the screws to 60±2 inch-pounds (7±.25 Newton-meters).

5. Install the AGILITY Cover on the back shell. Attach the head support.

Removal

1. Remove any attached head support. Remove the AGILITY Cover from the back shell.
2. Use a 4mm allen wrench to loosen and remove the two screws and washers. Remove the AGILITY UniTrack Rail.

Troubleshooting

For additional assistance, contact the equipment provider, distributor, or Customer Support.

The AGILITY UniTrack Rail does not fit or does not stay secure on the back shell.

- » Check the model. AG-HR-UT14 fits only AGILITY 14xx widths. AG-HR-UT fits all other AGILITY widths.
- » Check for damage to the product, including the screws.
- » Check to make sure there are no unapproved accessories attached to the UniTrack.

A component is damaged.

Contact Customer Support for a replacement.

Cleaning and Disinfecting


Warnings:

- Follow all manufacturer germicidal disinfectant safety guidelines.
- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. Before use by a different individual: clean, disinfect, and check the product for proper functioning.
- Clean the product regularly and when it becomes soiled.

To clean: Hand wash with a sponge, using soap and water. Rinse with clean water. Wipe dry with a clean cloth and then allow to air dry completely.

To disinfect: Wipe gently with a cloth dampened with a household disinfectant. Rinse with clean water. Wipe dry with a clean cloth and then allow to air dry completely.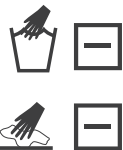

Recommendation: When washing by hand, use room-temperature water.

Care Symbols

Wipe with a damp cloth.

Hand wash.

Dry flat.

Storage, Transport, Disposal and Recycling

Storage or Transport: Clean and disinfect the product. Place in a container that protects the product from moisture, contaminants and damage. After storing or transporting, inspect the product to ensure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

Disposal: The components of the product are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incineration must be performed by a qualified, licensed waste management facility.

Recycling: Contact a local recycling agency to determine recycling options for the product.

Limited Warranty

Limited Warranty Term: 60 months; see also the Limited Warranty supplement provided with the product, or contact Customer Support.

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Permobil LTD