

Wheel Locks


Section 7



Parts Manual

COMPACT SCISSOR LOCK


ALL TiLite™ CHAIRS

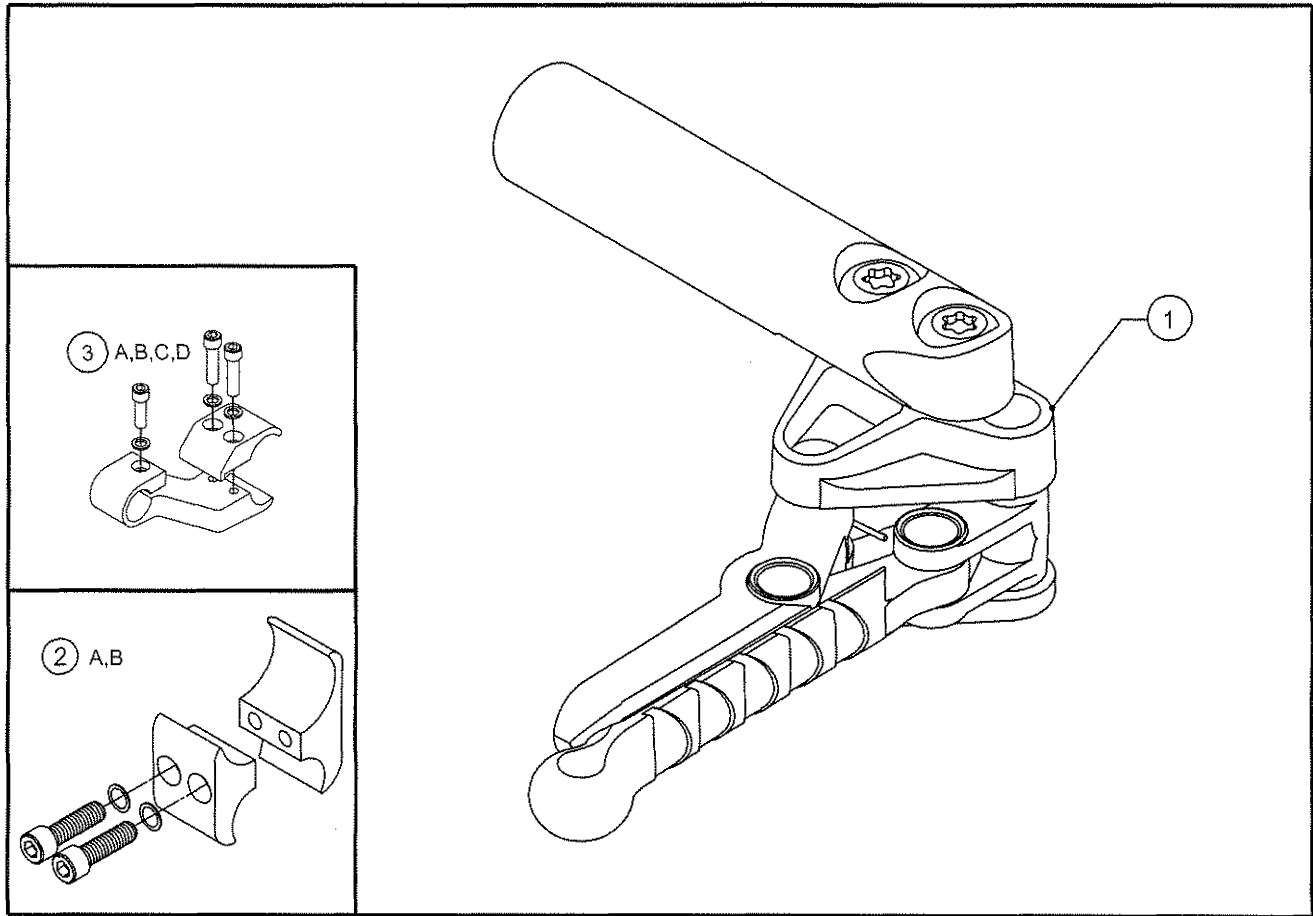

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS111502	COMPOSITE WHEEL LOCK, SCISSOR LOCK, RIGHT
1B	1	AS111503	COMPOSITE WHEEL LOCK, SCISSOR LOCK, LEFT
2A	2	CP100048	1" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
2B	2	AS102342	1-¼" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
3A	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
3B	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
3C	1	AS104423	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1¼" DIA TUBE)
3D	1	AS104424	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1¼" DIA TUBE)
1A,1B, 2A		AS111767	COMPOSITE SCISSOR LOCK W/ 1" CLAMP, PAIR
1A, 1B, 2B		AS111768	COMPOSITE SCISSOR LOCK W/ 1.25" CLAMP, PAIR
1A,1B,3A,3B		AS111769	COMPOSITE SCISSOR LOCK W/ 1" OFFSET CLAMP, PAIR
1A,1B,3C,3D		AS111770	COMPOSITE SCISSOR LOCK W/1.25" OFFSET CLAMP, PAIR

COMPACT PULL TO LOCK WHEEL LOCK

ALL TiLite™ CHAIRS

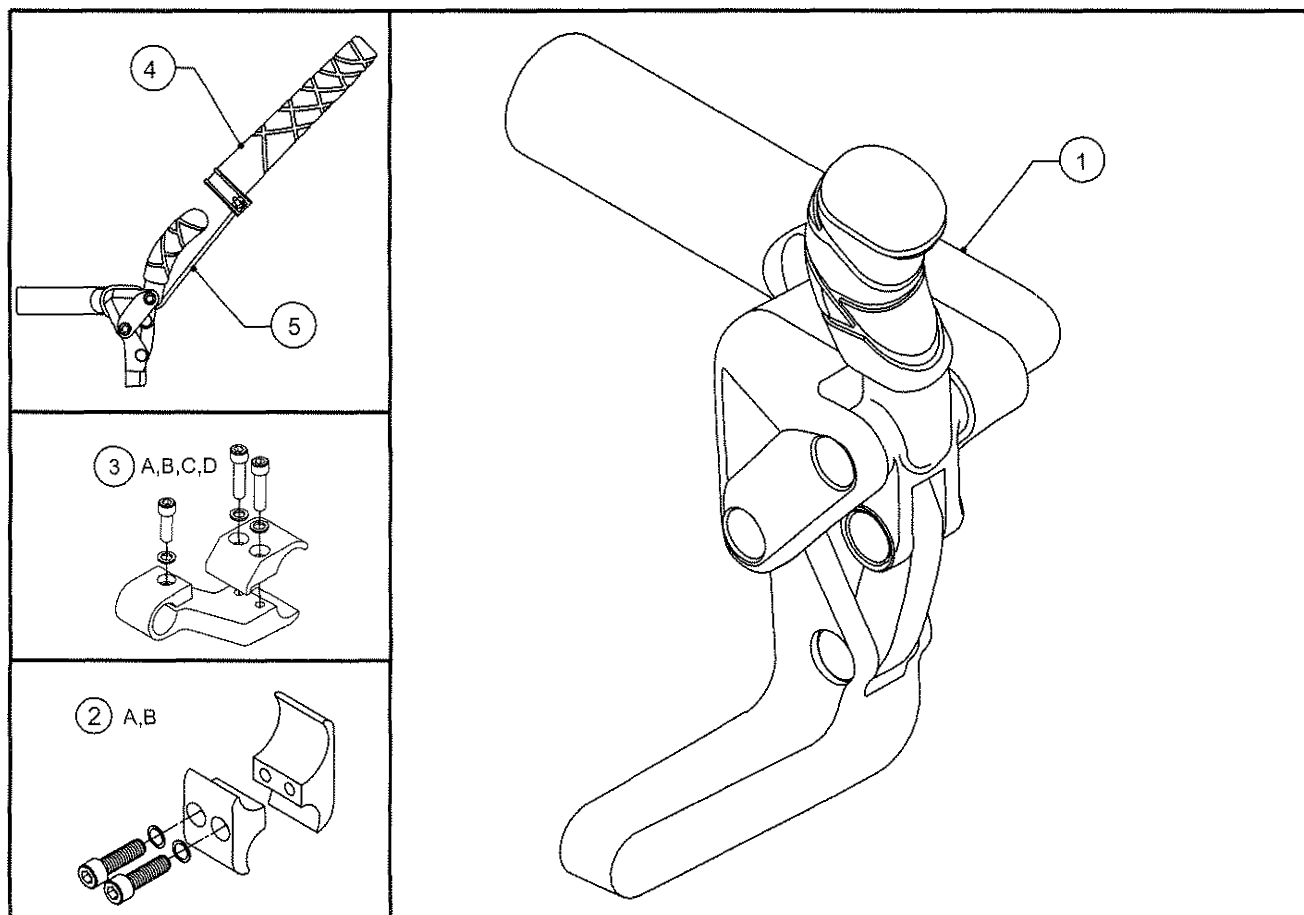


ITEM QUANTITY PART NO. DESCRIPTION

1A	1	AS111498	COMPOSITE WHEEL LOCK, PULL TO LOCK, RIGHT
1B	1	AS111499	COMPOSITE WHEEL LOCK, PULL TO LOCK, LEFT
2A	2	CP100048	1" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
2B	2	AS102342	1-¼" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
3A	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
3B	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
3C	1	AS104423	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1¼" DIA TUBE)
3D	1	AS104424	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1¼" DIA TUBE)
4	2	CP105920	WHEEL LOCK HANDLE EXTENTION, MOLDED
5	2	RW100214	1/8" SHOCK CORD, BLACK (10")
1A,1B, 2A		AS111771	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1" CLAMP, PAIR
1A, 1B, 2B		AS111772	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1.25" CLAMP, PAIR
1A,1B,3A,3B		AS111773	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1" OFFSET CLAMP, PAIR
1A,1B,3C,3D		AS111774	COMPOSITE WHEEL LOCK PULL TO LOCK W/1.25" OFFSET CLAMP, PAIR
1A,1B, 2A,4,5		AS112016	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1" CLAMP W/ EXT HANDLE, PAIR
1A, 1B, 2B,4,5		AS112017	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1.25" CLAMP W/ EXT HANDLE, PAIR
1A,1B,3A,3B,4,5		AS112018	COMP WHEEL LOCK PULL TO LOCK W/ 1" OFFSET CLAMP W/ EXT HANDLE, PAIR
1A,1B,3C,3D,4,5		AS112019	COMP WHEEL LOCK PULL TO LOCK W/1.25" OFFSET CLAMP W/ EXT HANDLE, PAIR

COMPACT PUSH TO LOCK WHEEL LOCK


ALL TiLite™ CHAIRS

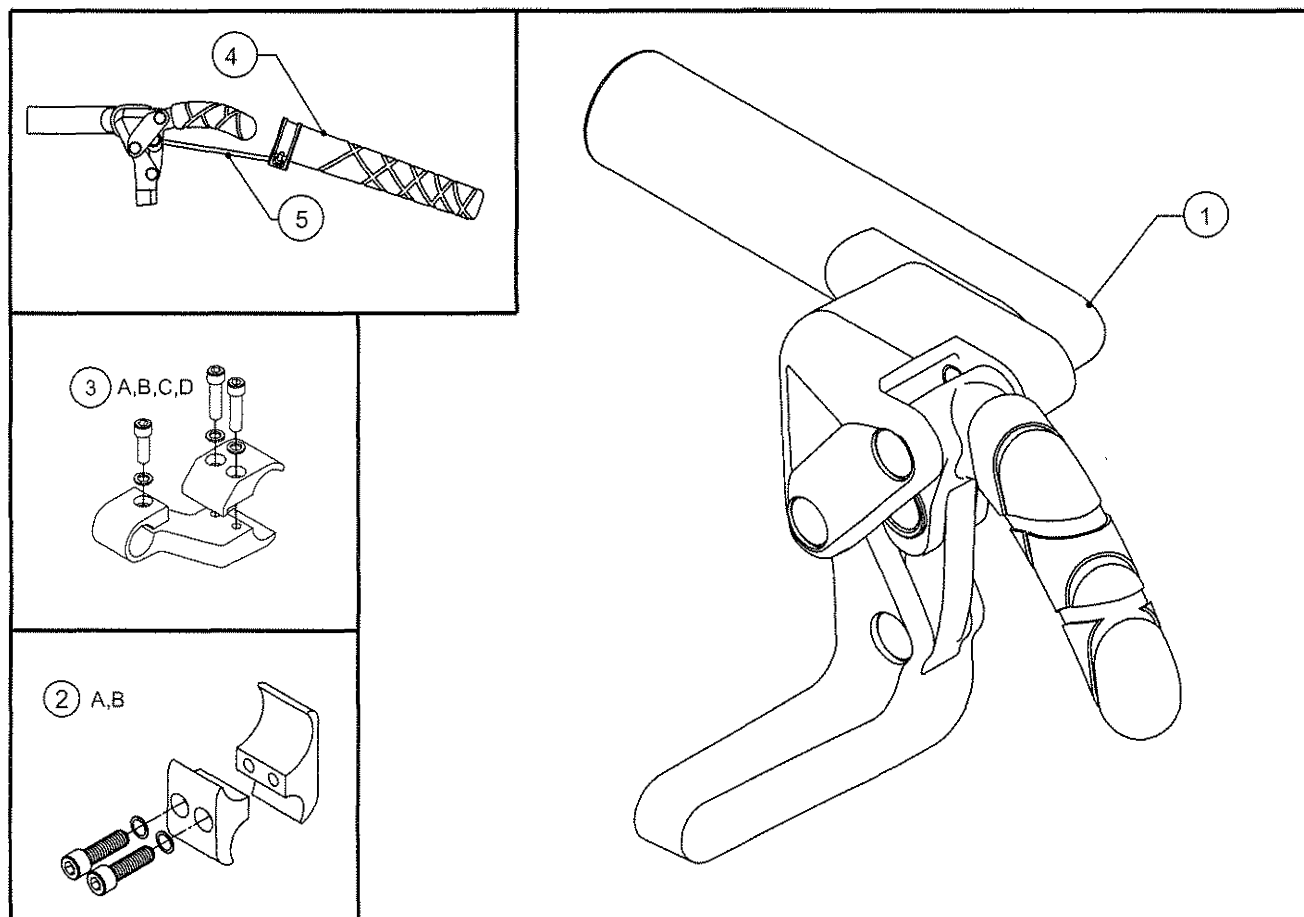

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS111500	COMPOSITE WHEEL LOCK, PUSH TO LOCK, RIGHT
1B	1	AS111501	COMPOSITE WHEEL LOCK, PUSH TO LOCK, LEFT
2A	2	CP100048	1" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
2B	2	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
3A	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
3B	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
3C	1	AS104423	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1 1/4" DIA TUBE)
3D	1	AS104424	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1 1/4" DIA TUBE)
4	2	CP105920	WHEEL LOCK HANDLE EXTENTION, MOLDED
5	2	RW100214	1/8" SHOCK CORD, BLACK (10")
1A,1B, 2A		AS111775	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1" CLAMP, PAIR
1A, 1B, 2B		AS111776	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1.25" CLAMP, PAIR
1A,1B,3A,3B		AS111777	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1" OFFSET CLAMP, PAIR
1A,1B,3C,3D		AS111778	COMPOSITE WHEEL LOCK PUSH TO LOCK W/1.25" OFFSET CLAMP, PAIR
1A,1B, 2A,4,5		AS112020	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1" CLAMP W/ EXT HANDLE, PAIR
1A, 1B, 2B,4,5		AS112021	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1.25" CLAMP W/ EXT HANDLE, PAIR
1A,1B,3A,3B,4,5		AS112022	COMP WHEEL LOCK PUSH TO LOCK W/ 1" OFFSET CLAMP W/ EXT HANDLE, PAIR
1A,1B,3C,3D,4,5		AS112023	COMP WHEEL LOCK PUSH TO LOCK W/1.25" OFFSET CLAMP W/ EXT HANDLE, PAIR

PUSH TO LOCK ALUM. SCISSOR LOCKS

SEE PRODUCTS AVAILABLE ON NEXT PAGE

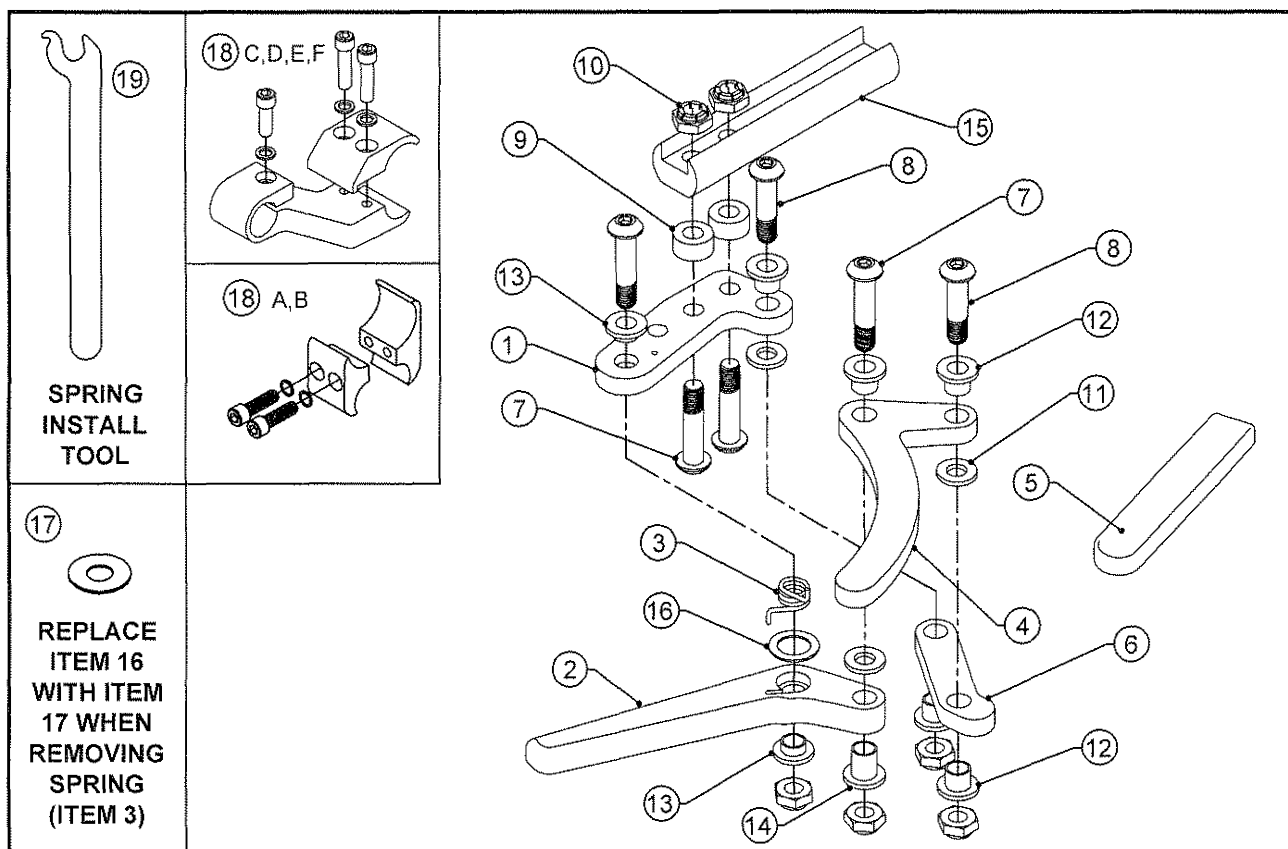

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103233	ALUMINUM SCISSOR LOCK MOUNT, RIGHT
1B	1	CP103234	ALUMINUM SCISSOR LOCK MOUNT, LEFT
2A	1	CP103231	ALUMINUM SCISSOR LOCK WHEELSTOP, RIGHT
2B	1	CP103232	ALUMINUM SCISSOR LOCK WHEELSTOP, LEFT
3A	1	CP102415	TORSION SPRING RIGHT HAND WHEELLOCK
3B	1	CP102416	TORSION SPRING LEFT HAND WHEELLOCK
4	1	CP103236	ALUMINUM SCISSOR LOCK HANDLE
5	1	HD100478	ALUMINUM SCISSOR LOCK GRIP
6	1	CP103235	ALUMINUM SCISSOR LOCK LINK
7	4	HD100476	1/4-28 X 1-1/8" X 3/8" THREAD BHSCS
8	2	HD100479	1/4-28 X 1" X 3/8" THREAD BHSCS
9	2	HD100016	NYLON STANDOFF .500" O.D. X .257" I.D. X .25 LONG
10	6	HD100520	1/4-28 X .219 TALL FLEX LOCK NUT BLACK ZINC PLATE
11	3	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
12	5	HD100481	FLANGED SCREW ISULATOR .260 ID X .250 LG
13	2	HD100482	FLANGED SCREW ISULATOR .260 ID X .125 LG
14	1	HD100474	FLANGED SCREW ISULATOR .260 ID X .375 LG
15	1	CP100180	WHEEL LOCK ADJUSTABLE MOUNT
16	1	HD100396	WASHER .418 ID X .630 OD X .030 THICK
17	1	HD100022	1/4 SAE FLAT WASHER, CLEAR ZINC PLATED
18A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
18B	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
18C	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
18D	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
18E	1	AS104423	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", LEFT (FITS 1 1/4" DIA TUBE)
18F	1	AS104424	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", RIGHT (FITS 1 1/4" DIA TUBE)
19	1	HD100390	SCISSORLOCK SPRING TOOL

SEE ASSEMBLIES ON NEXT PAGE

PUSH TO LOCK ALUM. SCISSOR LOCKS

SEE PRODUCTS AVAILABLE ON BELOW

FITS ALL CHAIRS LISTED BELOW

1A,2A,3A-17	AS104188	ALUMINUM SCISSOR PUSH TO LOCK, RIGHT ASSY (W/O CLAMP)
1B,2B,3B-17	AS104189	ALUMINUM SCISSOR PUSH TO LOCK, LEFT ASSY (W/O CLAMP)

FOR TR, TRC, YR, TRA, YRA, EVO, BB, YG, X, Aero X, 2GX, Aero R, TR (SERIES 2),

TR (SERIES 3), YR (SERIES 2), AERO T, TRA (SERIES 2)

1A,2A,3A-18A	AS104307	ALUMINUM SCISSOR PUSH TO LOCK, Ø1", RIGHT ASSY
1B,2B,3B-18A	AS104308	ALUMINUM SCISSOR PUSH TO LOCK, Ø1", LEFT ASSY


FOR ZR, ZRA, Aero Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

1A,2A,3A-18B	AS104309	ALUMINUM SCISSOR PUSH TO LOCK, Ø1-1/4", RIGHT ASSY
1B,2B,3B-18B	AS104310	ALUMINUM SCISSOR PUSH TO LOCK, Ø1-1/4", LEFT ASSY
1A,2A,3A-18E	AS104576	ALUMINUM SCISSOR PUSH TO LOCK, RIGHT W/ 1-1/4" OFFSET CLAMP
1B,2B,3B-18F	AS104575	ALUMINUM SCISSOR PUSH TO LOCK, LEFT W/ 1-1/4" OFFSET CLAMP



PULL TO LOCK ALUM. SCISSOR LOCKS

SEE PRODUCTS AVAILABLE ON NEXT PAGE

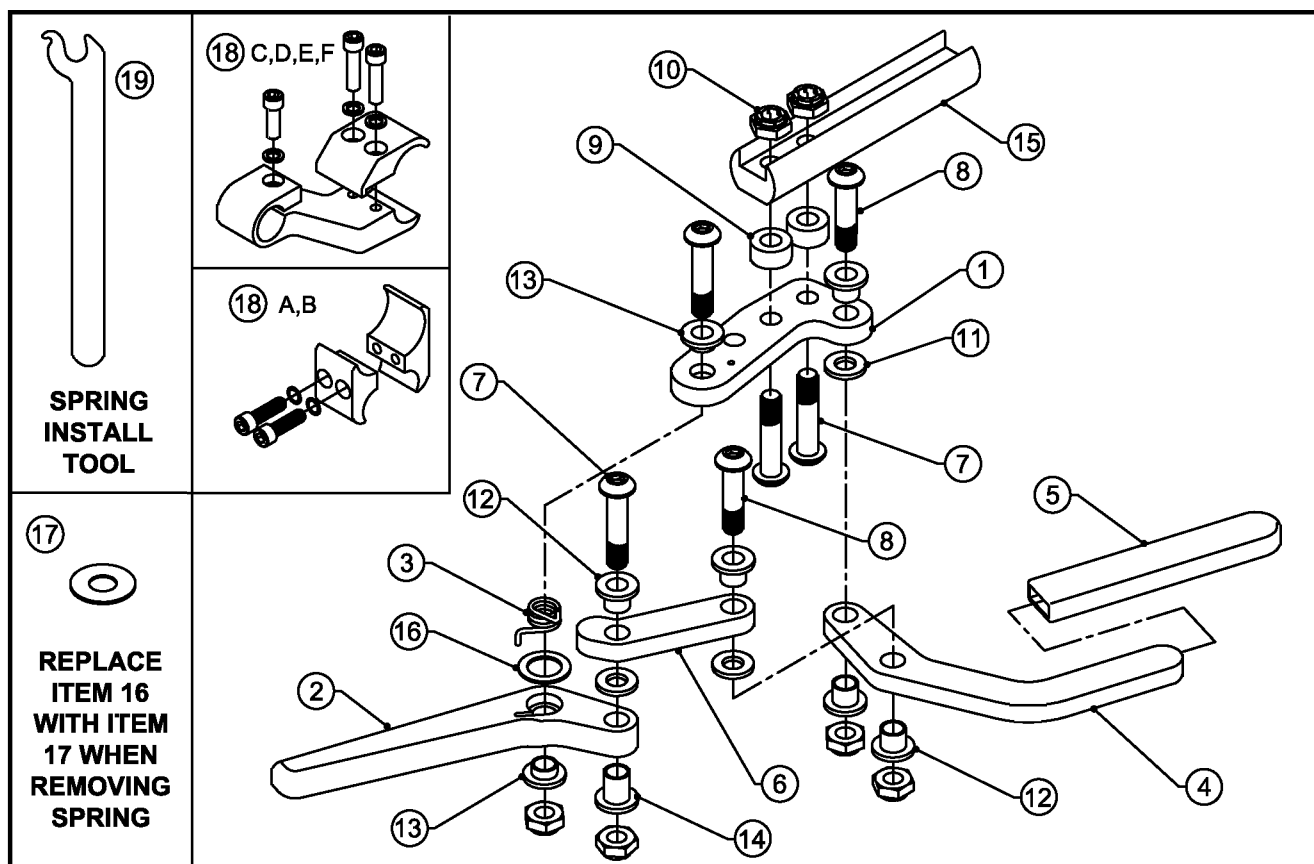

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103233	ALUMINUM SCISSOR LOCK MOUNT, RIGHT
1B	1	CP103234	ALUMINUM SCISSOR LOCK MOUNT, LEFT
2A	1	CP103231	ALUMINUM SCISSOR LOCK WHEELSTOP, RIGHT
2B	1	CP103232	ALUMINUM SCISSOR LOCK WHEELSTOP, LEFT
3A	1	CP102415	TORSION SPRING RIGHT HAND WHEELLOCK
3B	1	CP102416	TORSION SPRING LEFT HAND WHEELLOCK
4	1	CP103915	ALUMINUM SCISSOR LOCK PULL HANDLE
5	1	HD100478	ALUMINUM SCISSOR LOCK GRIP
6	1	CP103235	ALUMINUM SCISSOR LOCK LINK
7	4	HD100476	1/4-28 X 1-1/8" X 3/8" THREAD BHSCS
8	2	HD100479	1/4-28 X 1" X 3/8" THREAD BHSCS
9	2	HD100016	NYLON STANDOFF .500" O.D. X .257" I.D. X .25 LONG
10	6	HD100520	1/4-28 X .219 TALL FLEX LOCK NUT BLACK ZINC PLATE
11	3	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
12	5	HD100481	FLANGED SCREW ISULATOR .260 ID X .250 LG
13	2	HD100482	FLANGED SCREW ISULATOR .260 ID X .125 LG
14	1	HD100474	FLANGED SCREW ISULATOR .260 ID X .375 LG
15	1	CP100180	WHEEL LOCK ADJUSTABLE MOUNT
16	1	HD100396	WASHER .418 ID X .630 OD X .030 THICK
17	1	HD100022	1/4 SAE FLAT WASHER, CLEAR ZINC PLATED
18A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
18B	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
18C	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
18D	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
18E	1	AS104423	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", LEFT (FITS 1 1/4" DIA TUBE)
18F	1	AS104424	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", RIGHT (FITS 1 1/4" DIA TUBE)
19	1	HD100390	SCISSORLOCK SPRING TOOL

SEE ASSEMBLIES ON NEXT PAGE



Parts Manual

PULL TO LOCK ALUM. SCISSOR LOCKS

SEE PRODUCTS AVAILABLE ON BELOW

FITS ALL CHAIRS LISTED BELOW

1A,2A,3A-17	AS104545	ALUMINUM SCISSOR PULL TO LOCK, RIGHT (W/O CLAMP)
1B,2B,3B-17	AS104546	ALUMINUM SCISSOR PULL TO LOCK, LEFT (W/O CLAMP)

FOR TR, TRC, YR, TRA, YRA, EVO, BB, YG, Aero R, TR (SERIES 2),

TR (SERIES 3), YR (SERIES 2), AERO T, TRA (SERIES 2)

1A,2A,3A-18A	AS104426	ALUMINUM SCISSOR PULL TO LOCK, Ø1", RIGHT
1B,2B,3B-18A	AS104427	ALUMINUM SCISSOR PULL TO LOCK, Ø1", LEFT
1A,2A,3A-18C	AS104428	ALUMINUM SCISSOR PULL TO LOCK, Ø1", RIGHT W/ OFFSET CLAMP
1B,2B,3B-18D	AS104429	ALUMINUM SCISSOR PULL TO LOCK, Ø1", LEFT W/ OFFSET CLAMP


FOR ZR, ZRA, Aero Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

1A,2A,3A-18B	AS104430	ALUMINUM SCISSOR PULL TO LOCK, Ø1-1/4", RIGHT
1B,2B,3B-18B	AS104431	ALUMINUM SCISSOR PULL TO LOCK, Ø1-1/4", LEFT
1A,2A,3A-18E	AS104580	ALUMINUM SCISSOR PULL TO LOCK, LEFT W/ 1-1/4" OFFSET CLAMP
1B,2B,3B-18F	AS104581	ALUMINUM SCISSOR PULL TO LOCK, RIGHT W/ 1-1/4" OFFSET CLAMP



COMPOSITE SCISSOR LOCK

ALL TiLite™ CHAIRS

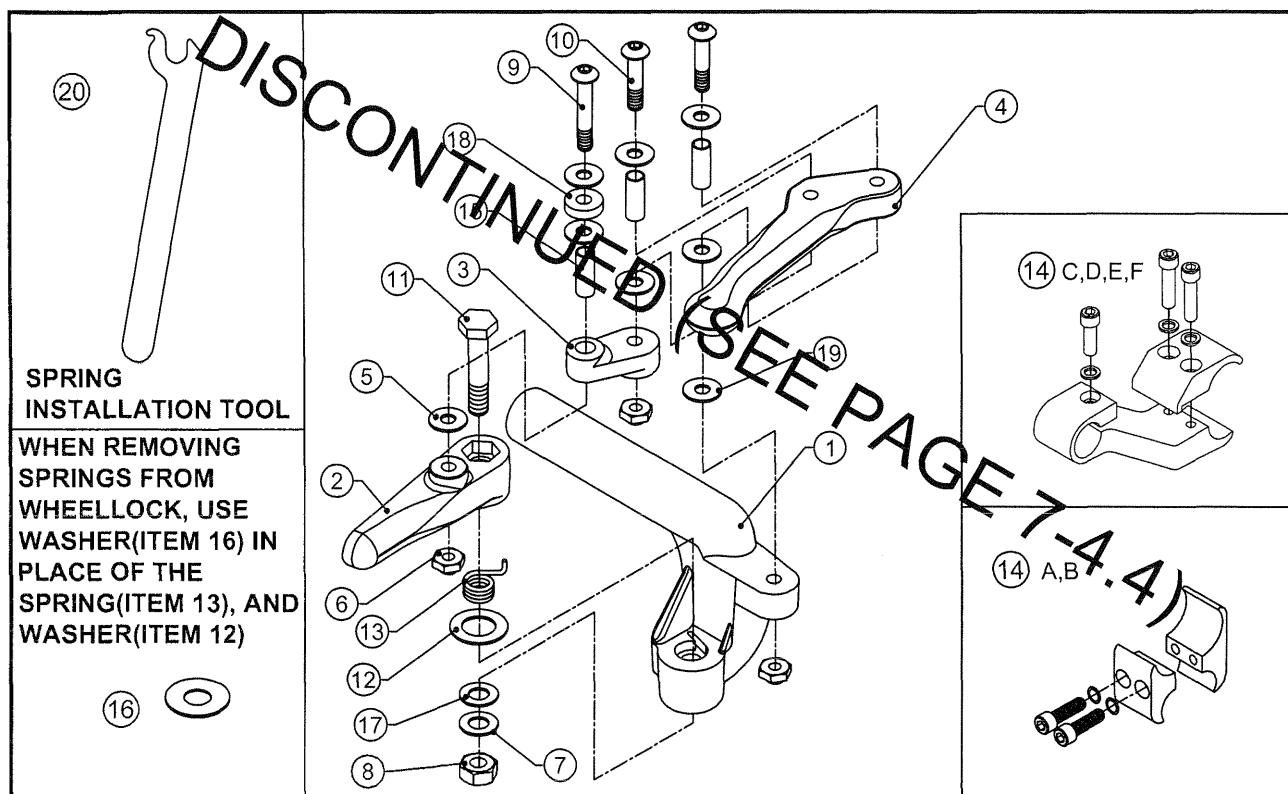

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP102479	MOUNT SCISSORLOCK COMPOSITE (RIGHT HAND)
1B	1	CP102480	MOUNT SCISSORLOCK COMPOSITE (LEFT HAND)
2A	1	CP102481	WHEELSTOP SCISSORLOCK COMPOSITE (RIGHT HAND)
2B	1	CP102482	WHEELSTOP SCISSORLOCK COMPOSITE (LEFT HAND)
3	1	CP102483	LINK SCISSORLOCK COMPOSITE
4	1	CP102484	HANDLE SCISSORLOCK COMPOSITE
5	7	HD100397	WASHER .203 ID X .375 OD X .060 THICK
6	3	HD100019	10-32 X .18 TALL FLEX LOCK NUT CLEAR ZINC PLATE
7	1	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
8	1	HD100077	1/4-28 TOP LOCK NUT GRADE C
9	1	HD100399	10-32 X 1 1/2" BUTTON HEAD CAP SCREW, BLACK ZINC
10	2	HD100400	10-32 X 1" BUTTON HEAD CAP SCREW, BLACK ZINC
11	1	HD100405	HEX BOLT AN4-15A, 1/4-28 X 1 5/8" LONG
12	1	HD100396	WASHER .418 ID X .630 OD X .030 THICK
13A	1	CP102415	TORSION SPRING RIGHT HAND WHEEL LOCK
13B	1	CP102416	TORSION SPRING LEFT HAND WHEEL LOCK
14A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
14B	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
14C	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
14D	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
14E	1	AS104423	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", LEFT (FITS 1 1/4" DIA TUBE)
14F	1	AS104424	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", RIGHT (FITS 1 1/4" DIA TUBE)
15	3	HD100388	SLEEVE .250 OD X .028 WALL X .400 LONG
16	2	HD100022	1/4 SAE FLAT WASHER, CLEAR ZINC PLATED
17	1	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, YELLOW ZINC
18	1	HD100236	NYLON SPACER .194 X .500 X .250
19	1	HD100401	WASHER .203 ID X .375 OD X .030 THICK
20	1	HD100390	SCISSORLOCK SPRING TOOL

COMPOSITE SCISSOR LOCK

SEE PRODUCTS AVAILABLE ON BELOW

1A,2A,3-12,13A,15-19
1B,2B,3-12,13B,15-19

AS102937
AS102938

COMPOSITE SCISSORLOCK RIGHT
COMPOSITE SCISSORLOCK LEFT

FOR TR, TRC, YR, BB, TRA, EVO, X, Aero X, 2GX, YG, Aero R, TR (SERIES 2), YR (SERIES 2)

1A,2A,3-12,13A,14A,15-19
1B,2B,3-12,13B,14A,15-19
1A,2A,3-12,13A,14D,15-19
1B,2B,3-12,13B,14C,15-19

AS102933
AS102934
AS103996
AS103995

COMPOSITE SCISSORLOCK RIGHT, 1"
COMPOSITE SCISSORLOCK LEFT, 1"
COMPOSITE SCISSORLOCK RIGHT, 1" W/ OFFSET CLAMP
COMPOSITE SCISSORLOCK LEFT, 1" W/ OFFSET CLAMP


FOR ZR, ZRA, Aero Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

1A,2A,3-12,13A,14B,15-19
1B,2B,3-12,13B,14B,15-19
1A,2A,3-12,13A,14E,15-19
1B,2B,3-12,13B,14F,15-19

AS102935
AS102936
AS104571
AS104572


COMPOSITE SCISSORLOCK RIGHT, 1¼"
COMPOSITE SCISSORLOCK LEFT, 1¼"
COMPOSITE SCISSOR LOCK LEFT, W/ 1-¼" OFFSET CLAMP
COMPOSITE SCISSOR LOCK RIGHT, W/ 1-¼" OFFSET CLAMP

DISCONTINUED (SEE PAGE 7-4.4)

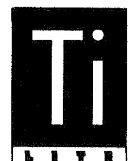

SHORT COMPOSITE SCISSOR LOCK

SEE PRODUCTS AVAILABLE ON NEXT PAGE

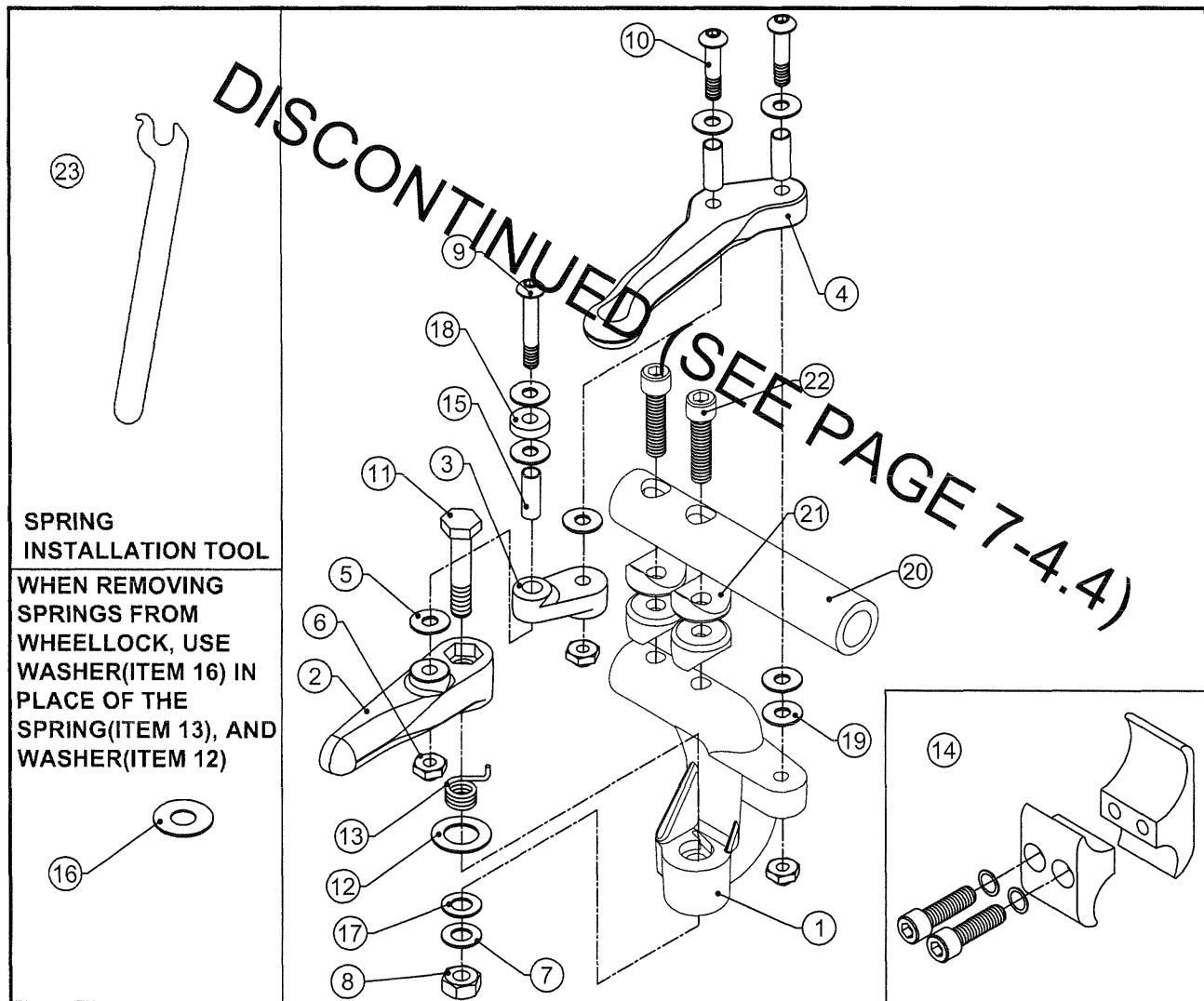

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103462	MOUNT SCISSORLOCK COMPOSITE SHORT (RIGHT HAND)
1B	1	CP103463	MOUNT SCISSORLOCK COMPOSITE SHORT (LEFT HAND)
2A	1	CP102481	MAKO WHEELSTOP RIGHT, COMPOSITE LOCK
2B	1	CP102482	MAKO WHEELSTOP LEFT, COMPOSITE LOCK
3	1	CP102483	LINK SCISSORLOCK COMPOSITE
4	1	CP102484	HANDLE SCISSORLOCK COMPOSITE
5	7	HD100397	WASHER .203 ID X .375 OD X .060 THICK
6	3	HD100019	10-32 X .18 TALL FLEX LOCK NUT CLEAR ZINC PLATE
7	1	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
8	1	HD100077	1/4-28 TOP LOCK NUT GRADE C
9	1	HD100399	10-32 X 1 1/2" BUTTON HEAD CAP SCREW, BLACK ZINC
10	2	HD100400	10-32 X 1" BUTTON HEAD CAP SCREW, BLACK ZINC
11	1	HD100405	HEX BOLT AN4-15A, 1/4-28 X 1 5/8" LONG
12	1	HD100396	WASHER .418 ID X .630 OD X .030 THICK
13A	1	CP102415	TORSION SPRING RIGHT HAND WHEEL LOCK
13B	1	CP102416	TORSION SPRING LEFT HAND WHEEL LOCK
14A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
14B	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
15	3	HD100388	SLEEVE .250 OD X .028 WALL X .400 LONG
16	2	HD100022	1/4 SAE FLAT WASHER, CLEAR ZINC PLATED
17	1	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, YELLOW ZINC
18	1	HD100236	NYLON SPACER .194 X .500 X .250
19	1	HD100401	WASHER .203 ID X .375 OD X .030 THICK
20	1	CP103464	COMPOSITE SCISSORLOCK EXTENSION TUBE
21	4	HD100119	COVED WASHER 3/4" OD X 1/4" ID X 1/8" THK
22	2	HD100051	1/4-20 X 1" SOCKET HEAD CAP SCREW, CLEAR ZINC
23	1	HD100390	SCISSORLOCK SPRING TOOL

ASSEMBLIES ON NEXT PAGE

7-3.2

SHORT COMPOSITE SCISSOR LOCK (CONT.)

SEE PRODUCTS AVAILABLE BELOW


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A,2A,3-12,13A,15-22		AS103704	SHORT COMPOSITE SCISSORLOCK RIGHT
1B,2B,3-12,13B,15-22		AS103705	SHORT COMPOSITE SCISSORLOCK LEFT
<u>FOR TR, TRC, YR, TRA, YRA, EVO, BB, YG, X, Aero X, 2GX, Aero R, TR (SERIES 2), YR (SERIES 2)</u>			
1A,2A,3-12,13A,14A,15-22		AS103706	SHORT COMPOSITE SCISSORLOCK RIGHT, 1"
1B,2B,3-12,13B,14A,15-22		AS103707	SHORT COMPOSITE SCISSORLOCK LEFT, 1"
<u>FOR ZR, ZRA, Aero Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)</u>			
1A,2A,3-12,13A,14B,15-22		AS103708	SHORT COMPOSITE SCISSORLOCK RIGHT, 1¼"
1B,2B,3-12,13B,14B,15-22		AS103709	SHORT COMPOSITE SCISSORLOCK LEFT, 1¼"

DISCONTINUED (SEE PAGE 7-4.4)



COMPOSITE SCISSOR LOCK, MAKO

SEE PRODUCTS AVAILABLE ON BELOW

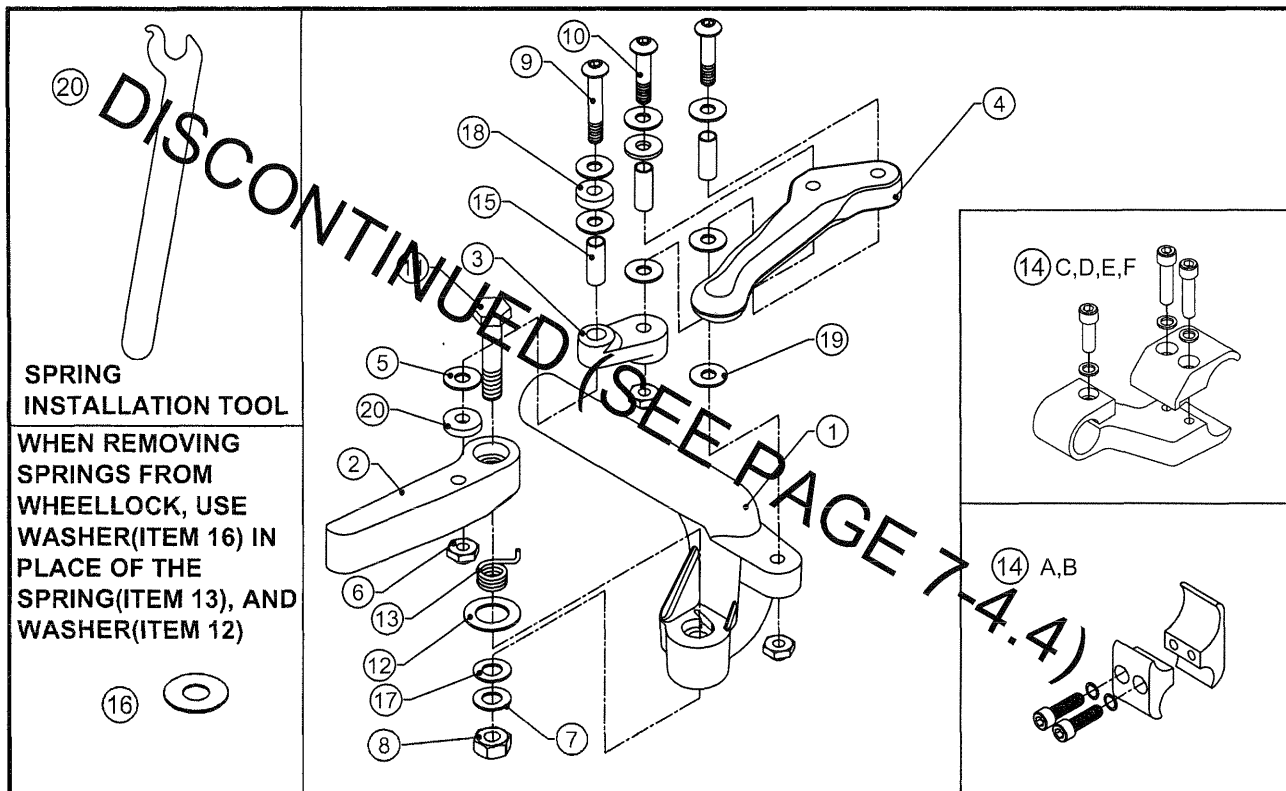

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP102479	MOUNT SCISSORLOCK COMPOSITE (RIGHT HAND)
1B	1	CP102480	MOUNT SCISSORLOCK COMPOSITE (LEFT HAND)
2A	1	CP102733	MAKO WHEELSTOP RIGHT, COMPOSITE LOCK
2B	1	CP102734	MAKO WHEELSTOP LEFT, COMPOSITE LOCK
3	1	CP102483	LINK SCISSORLOCK COMPOSITE
4	1	CP102484	HANDLE SCISSORLOCK COMPOSITE
5	7	HD100397	WASHER .203 ID X .375 OD X .060 THICK
6	3	HD100019	10-32 X .18 TALL FLEX LOCK NUT CLEAR ZINC PLATE
7	1	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
8	1	HD100077	1/4-28 TOP LOCK NUT GRADE C
9	1	HD100403	10-32 X 1.75" BUTTON HEAD CAP SCREW, BLACK ZINC
10	2	HD100400	10-32 X 1" BUTTON HEAD CAP SCREW, BLACK ZINC
11	1	HD100405	HEX BOLT AN4-15A, 1/4-28 X 1 5/8" LONG
12	1	HD100396	WASHER .418 ID X .630 OD X .030 THICK
13A	1	CP102415	TORSION SPRING RIGHT HAND WHEELLOCK
13B	1	CP102416	TORSION SPRING LEFT HAND WHEELLOCK
14A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
14B	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
14C	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
14D	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
14E	1	AS104423	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", LEFT (FITS 1 1/4" DIA TUBE)
14F	1	AS104424	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", RIGHT (FITS 1 1/4" DIA TUBE)
15	3	HD100388	SLEEVE .250 OD X .028 WALL X .400 LONG
16	2	HD100022	1/4 SAE FLAT WASHER, CLEAR ZINC PLATED
17	1	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, YELLOW ZINC
18	1	HD100498	WASHER NYLON .194 ID X .500 OD X .125 THK
19	2	HD100401	WASHER .203 ID X .375 OD X .030 THICK
20	1	HD100404	WASHER .203 ID X .309 OD X .125 THICK
21	1	HD100390	SCISSORLOCK SPRING TOOL

COMPOSITE SCISSOR LOCK, MAKO

SEE PRODUCTS AVAILABLE ON BELOW

1A,2A,3-12,13A,15-20
1B,2B,3-12,13B,15-20

AS103031
AS103032

COMPOSITE SCISSOR LOCK RIGHT, MAKO
COMPOSITE SCISSOR LOCK LEFT, MAKO

FOR TR, TRC, YR, BB, TRA, EVO, X, Aero X, 2GX, YG, Aero R, TR (SERIES 2), YR (SERIES 2)

1A,2A,3-12,13A,14A,15-20
1B,2B,3-12,13B,14A,15-20
1A,2A,3-12,13A,14D,15-20
1B,2B,3-12,13B,14C,15-20

AS103033
AS103034
AS103998
AS103997

COMPOSITE SCISSOR LOCK RIGHT, 1", MAKO
COMPOSITE SCISSOR LOCK LEFT, 1", MAKO
COMP SCISSOR LOCK RIGHT, 1", MAKO W/ OFFSET CLAMP
COMP SCISSOR LOCK LEFT, 1", MAKO W/ OFFSET CLAMP

FOR ZR, ZRA, Aero Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

1A,2A,3-12,13A,14B,15-20
1B,2B,3-12,13B,14B,15-20
1A,2A,3-12,13A,14F,15-20
1B,2B,3-12,13B,14E,15-20

AS103035
AS103036
AS104574
AS104573


COMPOSITE SCISSOR LOCK RIGHT, 1¼", MAKO
COMPOSITE SCISSOR LOCK LEFT, 1¼", MAKO
COMPOSITE SCISSOR LOCK RIGHT, POLY, W/ 1-¼" OFFSET CLAMP
COMPOSITE SCISSOR LOCK LEFT, POLY, W/ 1-¼" OFFSET CLAMP

DISCONTINUED (SEE PAGE 7-4.4)



SHORT COMPOSITE SCISSOR LOCK, MAKO

SEE PRODUCTS AVAILABLE ON NEXT PAGE

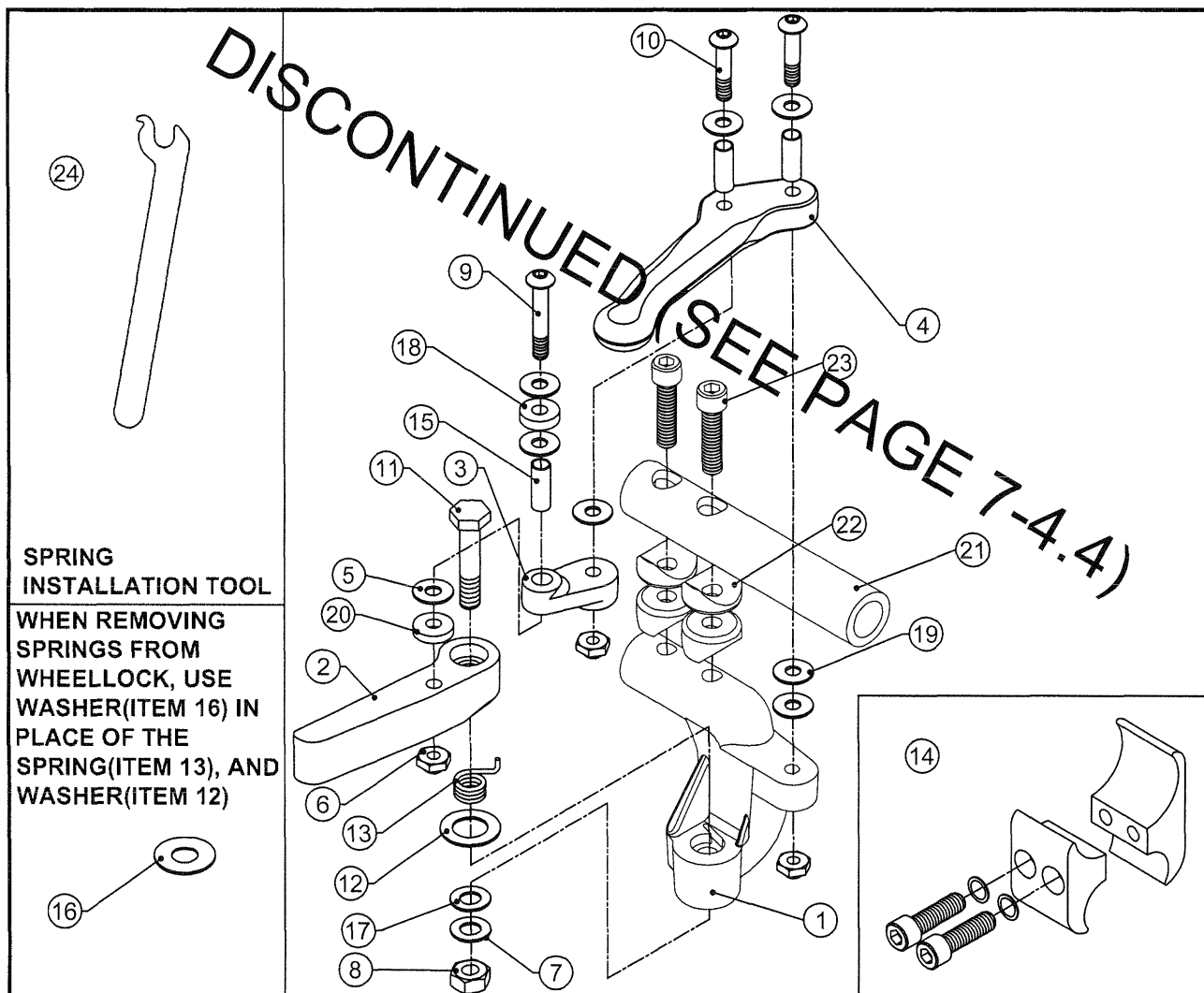

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103462	MOUNT SCISSORLOCK COMPOSITE SHORT (RIGHT HAND)
1B	1	CP103463	MOUNT SCISSORLOCK COMPOSITE SHORT (LEFT HAND)
2A	1	CP102733	MAKO WHEELSTOP RIGHT, COMPOSITE LOCK
2B	1	CP102734	MAKO WHEELSTOP LEFT, COMPOSITE LOCK
3	1	CP102483	LINK SCISSORLOCK COMPOSITE
4	1	CP102484	HANDLE SCISSORLOCK COMPOSITE
5	7	HD100397	WASHER .203 ID X .375 OD X .060 THICK
6	3	HD100019	10-32 X .18 TALL FLEX LOCK NUT CLEAR ZINC PLATE
7	1	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
8	1	HD100077	1/4-28 TOP LOCK NUT GRADE C
9	1	HD100403	10-32 X 1.75" BUTTON HEAD CAP SCREW, BLACK ZINC
10	2	HD100400	10-32 X 1" BUTTON HEAD CAP SCREW, BLACK ZINC
11	1	HD100405	HEX BOLT AN4-15A, 1/4-28 X 1 5/8" LONG
12	1	HD100396	WASHER .418 ID X .630 OD X .030 THICK
13A	1	CP102415	TORSION SPRING RIGHT HAND WHEEL LOCK
13B	1	CP102416	TORSION SPRING LEFT HAND WHEEL LOCK
14A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
14B	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
15	3	HD100388	SLEEVE .250 OD X .028 WALL X .400 LONG
16	2	HD100022	1/4 SAE FLAT WASHER, CLEAR ZINC PLATED
17	1	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, YELLOW ZINC
18	1	HD100498	WASHER NYLON .194 ID X .500 OD X .125 THK
19	2	HD100401	WASHER .203 ID X .375 OD X .030 THICK
20	1	HD100404	WASHER .203 ID X .309 OD X .125 THICK
21	1	CP103464	COMPOSITE SCISSORLOCK EXTENSION TUBE
22	4	HD100119	COVED WASHER 3/4" OD X 1/4" ID X 1/8" THK
23	2	HD100051	1/4-20 X 1" SOCKET HEAD CAP SCREW, CLEAR ZINC
24	1	HD100390	SCISSORLOCK SPRING TOOL


ASSEMBLIES ON NEXT PAGE

SHORT COMPOSITE SCISSOR LOCK, MAKO (CONT.)

SEE PRODUCTS AVAILABLE ON BELOW


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A,2A,3-12,13A,15-23		AS103710	SHORT COMPOSITE SCISSORLOCK RIGHT, MAKO
1B,2B,3-12,13B,15-23		AS103711	SHORT COMPOSITE SCISSORLOCK LEFT, MAKO
<u>FOR TR, TRC, YR, TRA, YRA, EVO, BB, YG, X, Aero X, 2GX, Aero R, TR (SERIES 2), YR (SERIES 2)</u>			
1A,2A,3-12,13A,14A,15-23		AS103712	SHORT COMPOSITE SCISSORLOCK RIGHT, 1", MAKO
1B,2B,3-12,13B,14A,15-23		AS103713	SHORT COMPOSITE SCISSORLOCK LEFT, 1", MAKO
<u>FOR ZR, ZRA, Aero Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)</u>			
1A,2A,3-12,13A,14B,15-23		AS103714	SHORT COMPOSITE SCISSORLOCK RIGHT, 1¼", MAKO
1B,2B,3-12,13B,14B,15-23		AS103715	SHORT COMPOSITE SCISSORLOCK LEFT, 1¼", MAKO

DISCONTINUED (SEE PAGE 7-4.4)

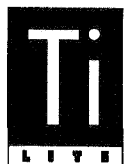

WHEEL LOCKS

PUSH/PULL, FITS ALL CHAIRS

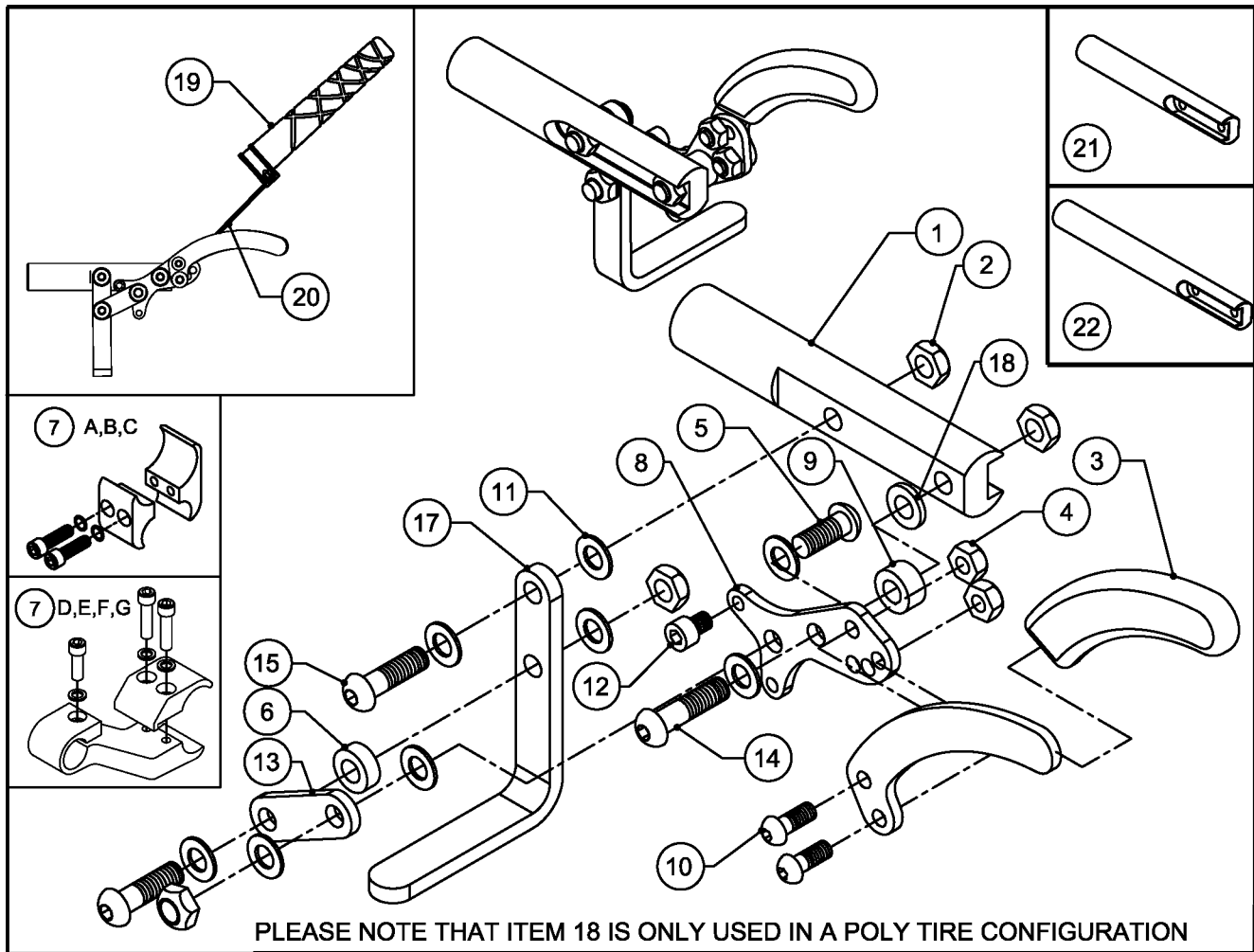

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100249	PUSH/PULL WHEEL LOCK ADJUSTABLE MOUNT
2	4	HD100077	1/4-28 TOP LOCK NUT GRADE C
3	1	HD100024	WHEEL LOCK FOAM HANDLE GRIP
4	2	HD100019	10-32 FLEX LOCK
5	1	HD100074	1/4-28 X 5/8 BUTTON HEAD CAP SCREW
6	1	HD100079	1/4 ID X 1/2 OD X 3/16 THICK NYLON SPACER
7A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
7B	1	AS102128	WHEEL LOCK CLAMP 3/4" TO 3/4"
7C	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
7D	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
7E	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
7F	1	AS104423	OFFSET WHEELLOCK ASSEMBLY 1-1/4", LEFT (FITS 1 1/4" DIA TUBE)
7G	1	AS104424	OFFSET WHEELLOCK ASSEMBLY 1-1/4", RIGHT (FITS 1 1/4" DIA TUBE)
8	1	CP100244	PUSH/PULL WHEEL LOCK HANDLE BASE
9	1	HD100016	1/4 ID X 1/2 OD X 1/4" NYLON SPACER
10	2	HD100009	10-32 X 1/2 BUTTON HEAD CAP SCREW
11	8	HD100067	1/4" OD X 1/2 OD X .031 THICK NYLON WASHER
12	1	HD100075	10-32 X 1/4 SOCKET HEAD CAP SCREW
13	1	CP100246	PUSH/PULL WHEEL LOCK LINK
14	1	HD100073	1/4-28 X 1 1/8 BUTTON HEAD CAP SCREW
15	2	HD100080	1/4-28 X 1" BUTTON HEAD CAP SCREW
16A	1	CP100247	PUSH/PULL HANDLE LEFT
16B	1	CP100262	PUSH/PULL HANDLE RIGHT
17A	1	CP100245	PUSH/PULL WHEEL LOCK STOP
17B	1	CP100773	PUSH/PULL WHEEL LOCK STOP RIGHT (POLY TIRE)
17C	1	CP100774	PUSH/PULL WHEEL LOCK STOP LEFT (POLY TIRE)
18	1	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK (POLY TIRE)
19	1	CP105920	WHEEL LOCK HANDLE EXTENTION, MOLDED
20	1	RW100214	1/8" SHOCK CORD, BLACK (8")
21	1	CP101293	PUSH/PULL WHEELLOCK MOUNT LONG
22	1	CP107800	PUSH/PULL WHEELLOCK MOUNT EXTRA LONG

WHEEL LOCKS (CONTINUED)

PUSH/PULL, FITS ALL CHAIRS

FOR CHAIRS WITH 1" DIAMETER FRAME TUBES

ITEM	QTY	PART NO.	DESCRIPTION
1-7A,8-16A,17A		AS117004	PUSH / PULL WHEEL LOCK LEFT
1-7A,8-16B,17A		AS117005	PUSH / PULL WHEEL LOCK RIGHT
1-7A,8-16A,17C,18		AS117006	PUSH / PULL WHEEL LOCK LEFT (POLY TIRES)
1-7A,8-16B,17B,18		AS117007	PUSH / PULL WHEEL LOCK RIGHT (POLY TIRES)
1-7A,8-16A,17A,19-20		AS112024	PUSH / PULL WHEEL LOCK LEFT W/ EXT HANDLE
1-7A,8-16B,17A,19-20		AS112025	PUSH / PULL WHEEL LOCK RIGHT W/ EXT HANDLE
1-7A,8-16A,17C-20		AS112026	PUSH / PULL WHEEL LOCK LEFT W/ EXT HANDLE (POLY TIRES)
1-7A,8-16B,17B-20		AS112027	PUSH / PULL WHEEL LOCK RIGHT W/ EXT HANDLE (POLY TIRES)
1-7D,8-16A,17A		AS103986	PUSH/PULL WHEEL LOCK LEFT W/ OFFSET CLAMP
1-7E,8-16B,17A		AS103988	PUSH/PULL WHEEL LOCK RIGHT W/ OFFSET CLAMP
1-7D,8-16A,17C,18		AS103987	PUSH/PULL WHEEL LOCK LEFT, W/ OFFSET CLAMP (POLY TIRES)
1-7E,8-16B,17B,18		AS103989	PUSH/PULL WHEEL LOCK RIGHT, W/ OFFSET CLAMP (POLY TIRES)
1-7D,8-16A,17A,19-20		AS112028	PUSH / PULL WHEEL LOCK LEFT W/ OFFSET CLAMP W/ EXT HANDLE
1-7E,8-16B,17A,19-20		AS112029	PUSH / PULL WHEEL LOCK RIGHT W/ OFFSET CLAMP W/ EXT HANDLE
1-7D,8-16A,17C-20		AS112030	PUSH / PULL LOCK LEFT W/ OFFSET CLAMP W/ EXT HANDLE (POLY TIRES)
1-7E,8-16B,17B-20		AS112031	PUSH / PULL LOCK RIGHT W/ OFFSET CLAMP W/ EXT HANDLE (POLY TIRES)

FOR CHAIRS WITH 1-1/4" DIAMETER FRAME TUBES

ITEM	QTY	PART NO.	DESCRIPTION
1-7C,8-16A,17A		AS102352	1-1/4" PUSH/PULL WHEEL LOCK LEFT
1-7C,8-16B,17A		AS102351	1-1/4" PUSH/PULL WHEEL LOCK RIGHT
1-7C,8-16A,17C,18		AS102354	1-1/4" PUSH/PULL WHEEL LOCK LEFT (POLY TIRES)
1-7C,8-16B,17B,18		AS102353	1-1/4" PUSH/PULL WHEEL LOCK RIGHT (POLY TIRES)
1-7C,8-16A,17A,19-20		AS112032	1-1/4" PUSH/PULL WHEEL LOCK LEFT W/ EXT HANDLE
1-7C,8-16B,17A,19-20		AS112033	1-1/4" PUSH/PULL WHEEL LOCK RIGHT W/ EXT HANDLE
1-7C,8-16A,17C-20		AS112034	1-1/4" PUSH/PULL WHEEL LOCK LEFT W/ EXT HANDLE (POLY TIRES)
1-7C,8-16B,17B-20		AS112035	1-1/4" PUSH/PULL WHEEL LOCK RIGHT W/ EXT HANDLE (POLY TIRES)
1-7F,8-16A,17A,19-20		AS104562	PUSH/PULL WHEEL LOCK LEFT W/ 1-1/4" OFFSET CLAMP
1-7G,8-16B,17A,19-20		AS104563	PUSH/PULL WHEEL LOCK RIGHT W/ 1-1/4" OFFSET CLAMP
1-7F,8-16A,17C,18		AS104564	PUSH/PULL WHEEL LOCK LEFT, W/ 1-1/4" OFFSET CLAMP (POLY TIRES)
1-7G,8-16B,17B,18		AS104565	PUSH/PULL WHEEL LOCK RIGHT, W/ 1-1/4" OFFSET CLAMP (POLY TIRES)
1-7F,8-16A,17A,19-20		AS112036	PUSH / PULL WHEEL LOCK LEFT W/ 1-1/4" OFFSET CLAMP W/ EXT HANDLE
1-7G,8-16B,17A,19-20		AS112037	PUSH / PULL WHEEL LOCK RIGHT W/ 1-1/4" OFFSET CLAMP W/ EXT HANDLE
1-7F,8-16A,17C-20		AS112038	PUSH / PULL LOCK LEFT W/ 1-1/4" OFFSET CLAMP W/ EXT HANDLE (POLY TIRES)
1-7G,8-16B,17B-20		AS112039	PUSH / PULL LOCK RIGHT W/ 1-1/4" OFFSET CLAMP W/ EXT HANDLE (POLY TIRES)

FOR CHAIRS WITHOUT CLAMPS


ITEM	QTY	PART NO.	DESCRIPTION
1-6,8-16A,17A		AS100060	PUSH / PULL WHEEL LOCK LEFT WITHOUT CLAMPS
1-6,8-16B,17A		AS100061	PUSH / PULL WHEEL LOCK RIGHT WITHOUT CLAMPS
1-6,8-16A,17C,18		AS100060M	PUSH / PULL WHEEL LOCK LEFT (POLY TIRES) WITHOUT CLAMPS
1-6,8-16B,17B,18		AS100061M	PUSH / PULL WHEEL LOCK RIGHT (POLY TIRES) WITHOUT CLAMPS
1-6,8-16A,17A,19-20		AS117008	PUSH / PULL WHEEL LOCK LEFT W/ EXT HANDLE WITHOUT CLAMPS
1-6,8-16B,17A,19-20		AS117009	PUSH / PULL WHEEL LOCK RIGHT W/ EXT HANDLE WITHOUT CLAMPS
1-6,8-16A,17C-20		AS117010	PUSH / PULL WHEEL LOCK LEFT W/ EXT HANDLE (POLY TIRES) WITHOUT CLAMPS
1-6,8-16B,17B-20		AS117011	PUSH / PULL WHEEL LOCK RIGHT W/ EXT HANDLE (POLY TIRES) WITHOUT CLAMPS



Parts Manual

PUSH/PULL WHEEL LOCKS W/ EXT. HANDLE

FOR ALL TiLite™ CHAIRS

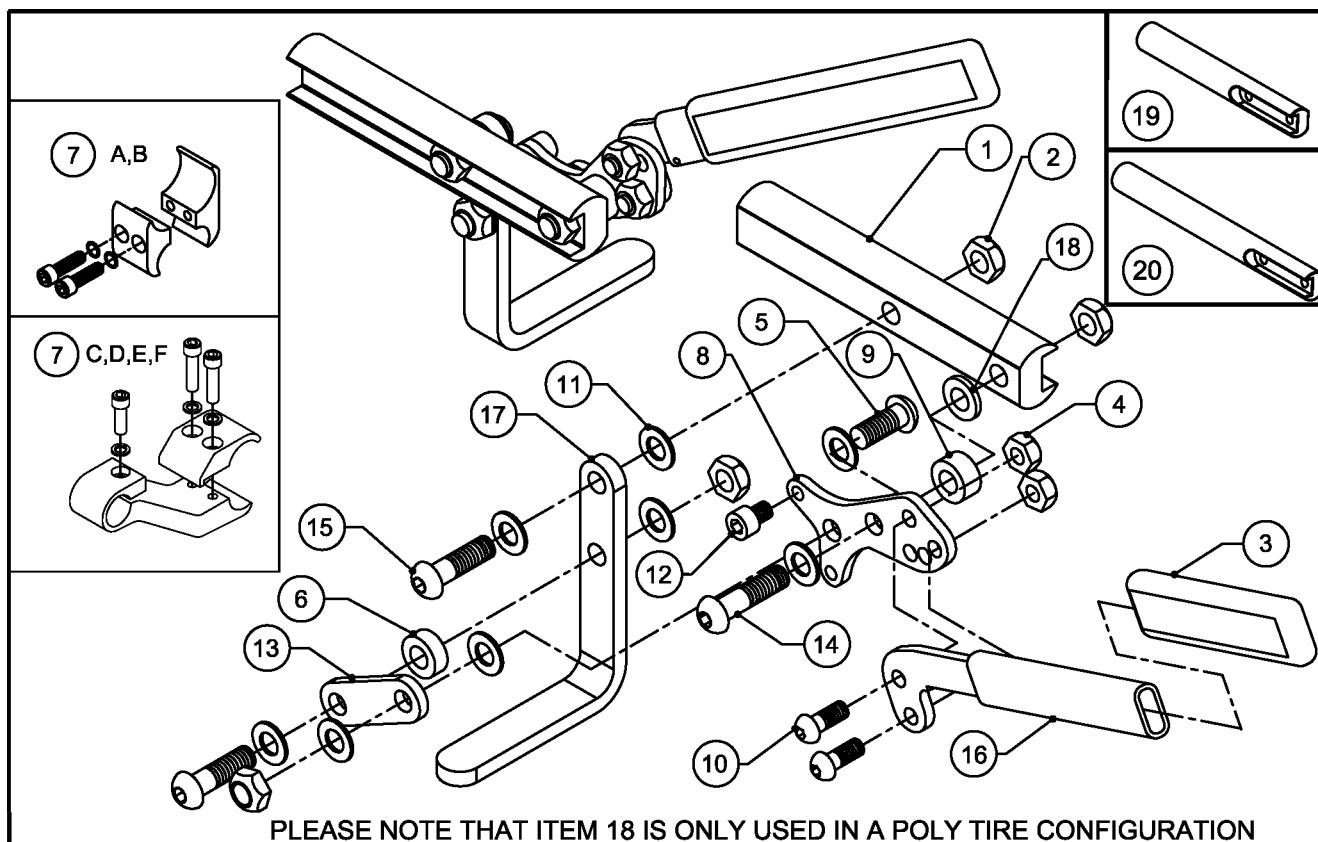

PLEASE NOTE THAT ITEM 18 IS ONLY USED IN A POLY TIRE CONFIGURATION

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP100249	PUSH/PULL WHEEL LOCK ADJUSTABLE MOUNT
2	4	HD100077	1/4-28 TOP LOCK NUT GRADE C
3	1	HD100266	GRIP EXTENSION HANDLE .25 X .615 X 4.00 SQUARE BLACK
4	2	HD100019	10-32 FLEX LOCK
5	1	HD100074	1/4-28 X 5/8 BUTTON HEAD CAP SCREW
6	1	HD100079	1/4 ID X 1/2 OD X 3/16 THICK NYLON SPACER
7A	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
7B	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
7C	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
7D	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
7E	1	AS104423	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", LEFT (FITS 1 1/4" DIA TUBE)
7F	1	AS104424	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", RIGHT (FITS 1 1/4" DIA TUBE)
8	1	CP100244	PUSH/PULL WHEEL LOCK HANDLE BASE
9	1	HD100016	1/4 ID X 1/2 OD X 1/4" NYLON SPACER
10	2	HD100009	10-32 X 1/2 BUTTON HEAD CAP SCREW
11	8	HD100067	1/4" OD X 1/2 OD X .031 THICK NYLON WASHER
12	1	HD100075	10-32 X 1/4 SOCKET HEAD CAP SCREW
13	1	CP100246	PUSH/PULL WHEEL LOCK LINK
14	1	HD100073	1/4-28 X 1 1/8 BUTTON HEAD CAP SCREW
15	2	HD100080	1/4-28 X 1" BUTTON HEAD CAP SCREW
16A	1	AS100963	EXTENSION HANDLE ASSEMBLY LEFT
16B	1	AS100964	EXTENSION HANDLE ASSEMBLY RIGHT
17A	1	CP100245	PUSH/PULL WHEEL LOCK STOP
17B	1	CP100773	PUSH/PULL WHEEL LOCK STOP RIGHT (POLY TIRE)
17C	1	CP100774	PUSH/PULL WHEEL LOCK STOP LEFT (POLY TIRE)
18	1	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK (POLY TIRE)
19	1	CP101293	PUSH/PULL WHEELLOCK MOUNT LONG
20	1	CP107800	PUSH/PULL WHEELLOCK MOUNT EXTRA LONG

PUSH/PULL WHEEL LOCKS W/ EXT. HANDLE


FOR ALL TiLite™ CHAIRS

1-7C,8-15,16A,17A	AS103990	PUSH/PULL WHEEL LOCK W/EXT. HANDLE LEFT, OFFSET
1-7D,8-15,16B,17A	AS103992	PUSH/PULL WHEEL LOCK W/EXT. HANDLE RIGHT, OFFSET
1-7C,8-15,16A,17C,18	AS103991	PUSH/PULL WHEEL LOCK W/EXT. HANDLE LEFT MAKO, OFFSET
1-7D,8-15,16B,17B,18	AS103993	PUSH/PULL WHEEL LOCK W/EXT. HANDLE RIGHT MAKO, OFFSET
1-7A,8-15,16A,17A	AS100965	PUSH/PULL WHEEL LOCK W/EXT. HANDLE LEFT
1-7A,8-15,16B,17A	AS100966	PUSH/PULL WHEEL LOCK W/EXT. HANDLE RIGHT
1-7A,8-15,16A,17C,18	AS100965M	PUSH/PULL WHEEL LOCK W/EXT. HANDLE LEFT MAKO
1-7A,8-15,16B,17B,18	AS100966M	PUSH/PULL WHEEL LOCK W/EXT. HANDLE RIGHT MAKO
1-7B,8-15,16A,17A	AS102363	1-¼" PUSH/PULL WHEEL LOCK W/ EXTENSION RIGHT
1-7B,8-15,16B,17A	AS102364	1-¼" PUSH/PULL WHEEL LOCK W/ EXTENSION LEFT
1-7B,8-15,16A,17C,18	AS102365	1-¼" PUSH/PULL WHEEL LOCK W/ EXT. POLY RIGHT
1-7B,8-15,16B,17B,18	AS102366	1-¼" PUSH/PULL WHEEL LOCK W/ EXT. POLY LEFT
1-7E,8-15,16A,17A	AS104566	PUSH/PULL WHEEL LOCK W/ EXT. LEFT, W/ 1-¼" OFFSET CLAMP
1-7F,8-15,16B,17A	AS104567	PUSH/PULL WHEEL LOCK W/ EXT. RIGHT, W/ 1-¼" OFFSET CLAMP
1-7E,8-15,16A,17C,18	AS104568	PUSH/PULL WHEEL LOCK, POLY W/ EXT. LEFT, W/ 1-¼" OFFSET CLAMP
1-7F,8-15,16B,17B,18	AS104569	PUSH/PULL WHEEL LOCK, POLY W/ EXT. RIGHT, W/ 1-¼" OFFSET CLAMP



COMPACT SCISSOR LOCK


ALL TiLite™ CHAIRS

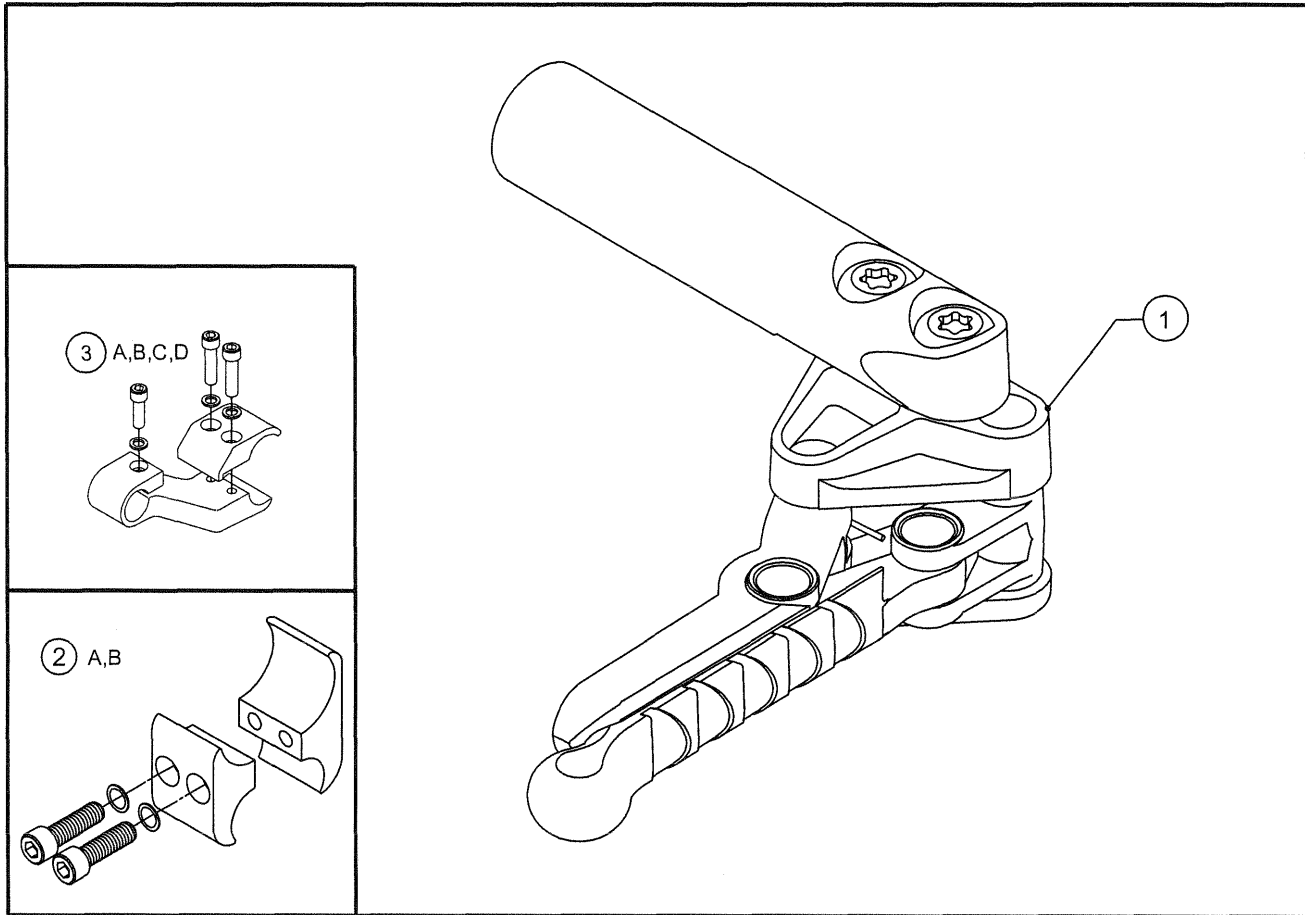

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS111502	COMPOSITE WHEEL LOCK, SCISSOR LOCK, RIGHT
1B	1	AS111503	COMPOSITE WHEEL LOCK, SCISSOR LOCK, LEFT
2A	2	CP100048	1" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
2B	2	AS102342	1-¼" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
3A	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
3B	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
3C	1	AS104423	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1¼" DIA TUBE)
3D	1	AS104424	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1¼" DIA TUBE)
1A,1B, 2A		AS111767	COMPOSITE SCISSOR LOCK W/ 1" CLAMP, PAIR
1A, 1B, 2B		AS111768	COMPOSITE SCISSOR LOCK W/ 1.25" CLAMP, PAIR
1A,1B,3A,3B		AS111769	COMPOSITE SCISSOR LOCK W/ 1" OFFSET CLAMP, PAIR
1A,1B,3C,3D		AS111770	COMPOSITE SCISSOR LOCK W/1.25" OFFSET CLAMP, PAIR

COMPACT PULL TO LOCK WHEEL LOCK


ALL TiLite™ CHAIRS

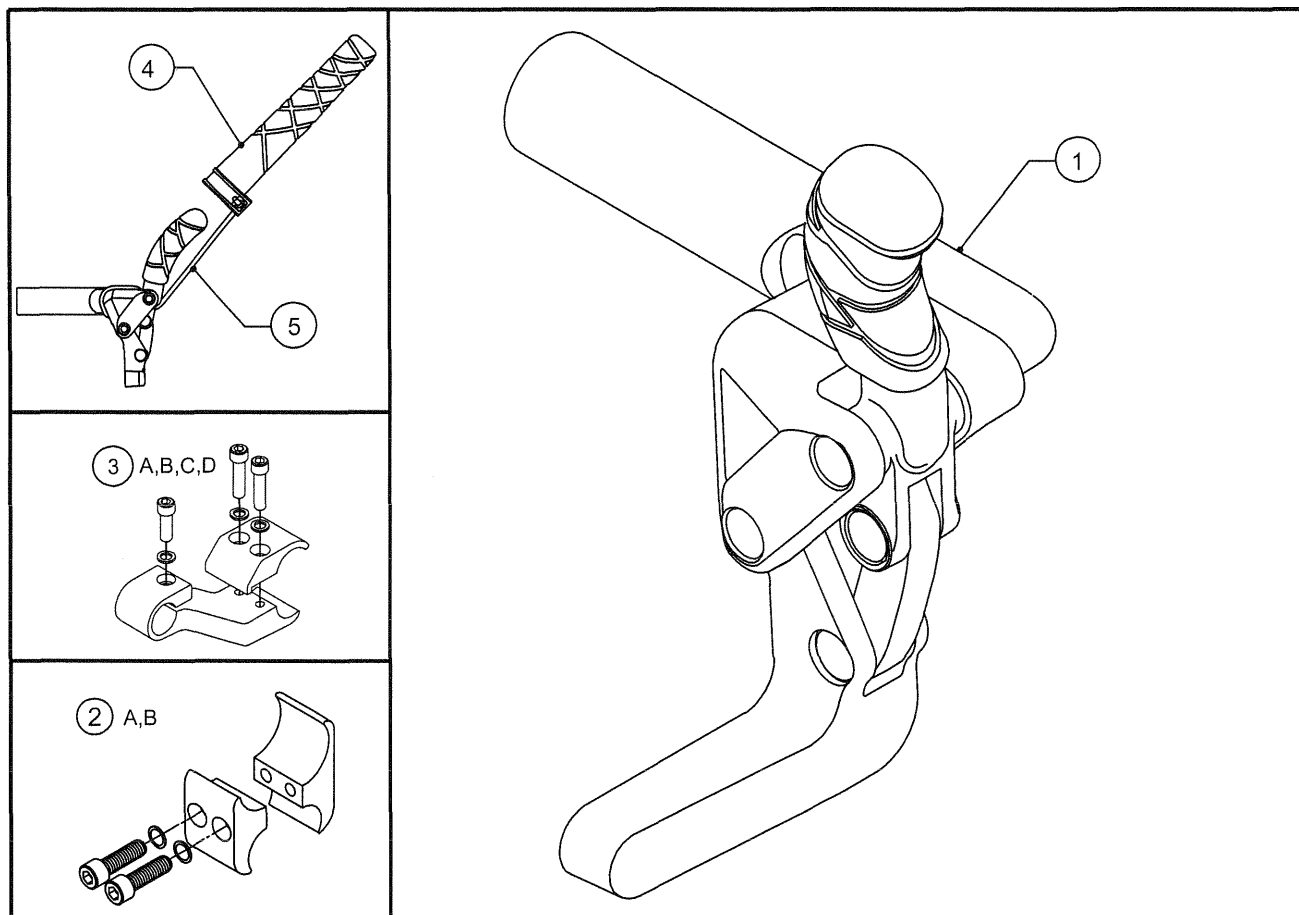

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS111498	COMPOSITE WHEEL LOCK, PULL TO LOCK, RIGHT
1B	1	AS111499	COMPOSITE WHEEL LOCK, PULL TO LOCK, LEFT
2A	2	CP100048	1" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
2B	2	AS102342	1-¼" TO ¾" WHEEL LOCK CLAMP ASSEMBLY
3A	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
3B	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
3C	1	AS104423	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1¼" DIA TUBE)
3D	1	AS104424	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1¼" DIA TUBE)
4	2	CP105920	WHEEL LOCK HANDLE EXTENTION, MOLDED
5	2	RW100214	1/8" SHOCK CORD, BLACK (10")
1A,1B, 2A		AS111771	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1" CLAMP, PAIR
1A, 1B, 2B		AS111772	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1.25" CLAMP, PAIR
1A,1B,3A,3B		AS111773	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1" OFFSET CLAMP, PAIR
1A,1B,3C,3D		AS111774	COMPOSITE WHEEL LOCK PULL TO LOCK W/1.25" OFFSET CLAMP, PAIR
1A,1B, 2A,4,5		AS112016	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1" CLAMP W/ EXT HANDLE, PAIR
1A, 1B, 2B,4,5		AS112017	COMPOSITE WHEEL LOCK PULL TO LOCK W/ 1.25" CLAMP W/ EXT HANDLE, PAIR
1A,1B,3A,3B,4,5		AS112018	COMP WHEEL LOCK PULL TO LOCK W/ 1" OFFSET CLAMP W/ EXT HANDLE, PAIR
1A,1B,3C,3D,4,5		AS112019	COMP WHEEL LOCK PULL TO LOCK W/1.25" OFFSET CLAMP W/ EXT HANDLE, PAIR

COMPACT PUSH TO LOCK WHEEL LOCK


ALL TiLite™ CHAIRS

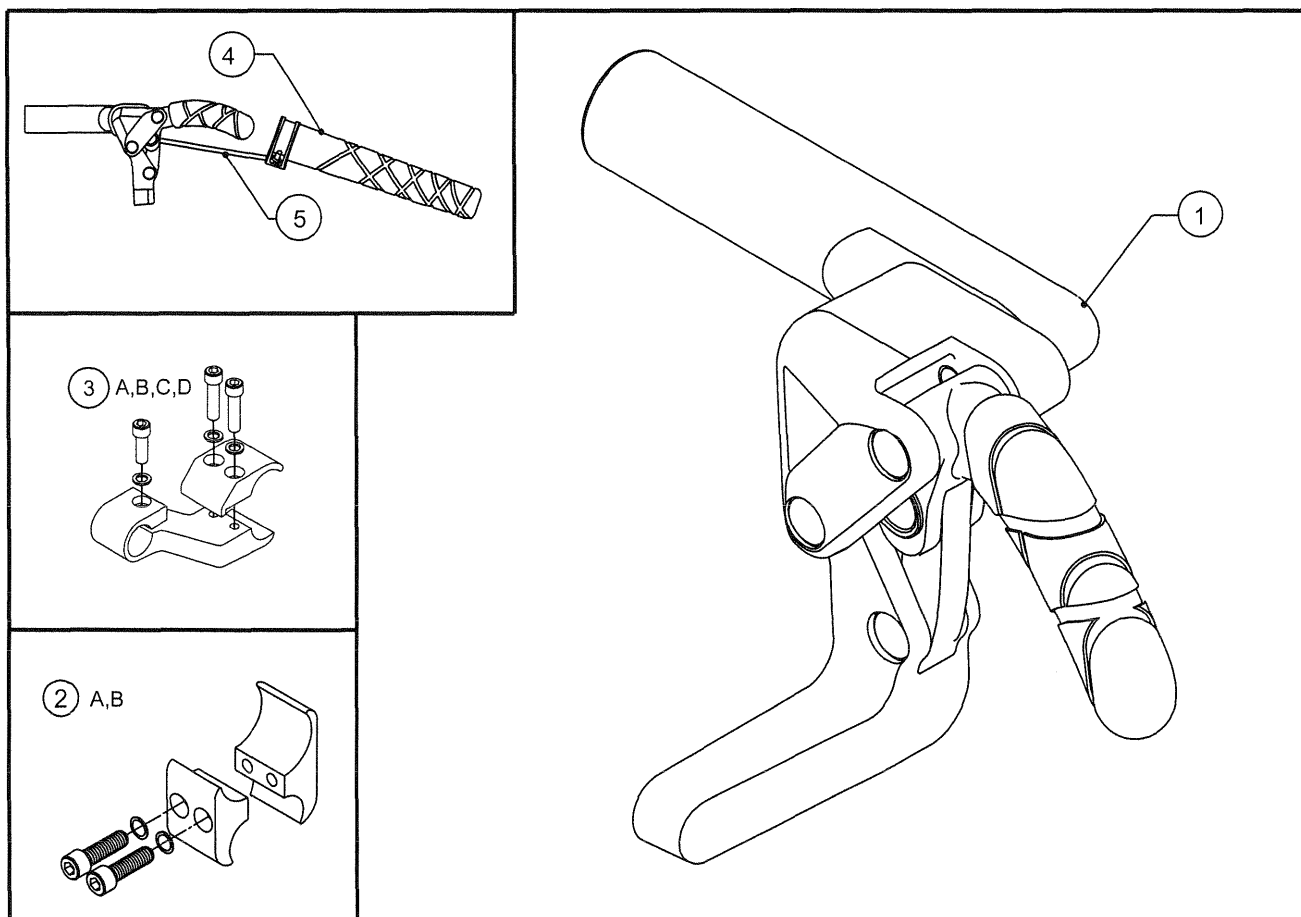

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS111500	COMPOSITE WHEEL LOCK, PUSH TO LOCK, RIGHT
1B	1	AS111501	COMPOSITE WHEEL LOCK, PUSH TO LOCK, LEFT
2A	2	CP100048	1" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
2B	2	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
3A	1	AS103681	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1" DIA TUBE)
3B	1	AS103683	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1" DIA TUBE)
3C	1	AS104423	OFFSET WHEEL LOCK CLAMP, LEFT (FITS 1 1/4" DIA TUBE)
3D	1	AS104424	OFFSET WHEEL LOCK CLAMP, RIGHT (FITS 1 1/4" DIA TUBE)
4	2	CP105920	WHEEL LOCK HANDLE EXTENTION, MOLDED
5	2	RW100214	1/8" SHOCK CORD, BLACK (10")
1A,1B, 2A		AS111775	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1" CLAMP, PAIR
1A, 1B, 2B		AS111776	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1.25" CLAMP, PAIR
1A,1B,3A,3B		AS111777	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1" OFFSET CLAMP, PAIR
1A,1B,3C,3D		AS111778	COMPOSITE WHEEL LOCK PUSH TO LOCK W/1.25" OFFSET CLAMP, PAIR
1A,1B, 2A,4,5		AS112020	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1" CLAMP W/ EXT HANDLE, PAIR
1A, 1B, 2B,4,5		AS112021	COMPOSITE WHEEL LOCK PUSH TO LOCK W/ 1.25" CLAMP W/ EXT HANDLE, PAIR
1A,1B,3A,3B,4,5		AS112022	COMP WHEEL LOCK PUSH TO LOCK W/ 1" OFFSET CLAMP W/ EXT HANDLE, PAIR
1A,1B,3C,3D,4,5		AS112023	COMP WHEEL LOCK PUSH TO LOCK W/1.25" OFFSET CLAMP W/ EXT HANDLE, PAIR

WHEEL LOCKS (DISCONTINUED)

SCISSOR LOCKS FITS ALL CHAIRS, EXCEPT TiLite™ SX

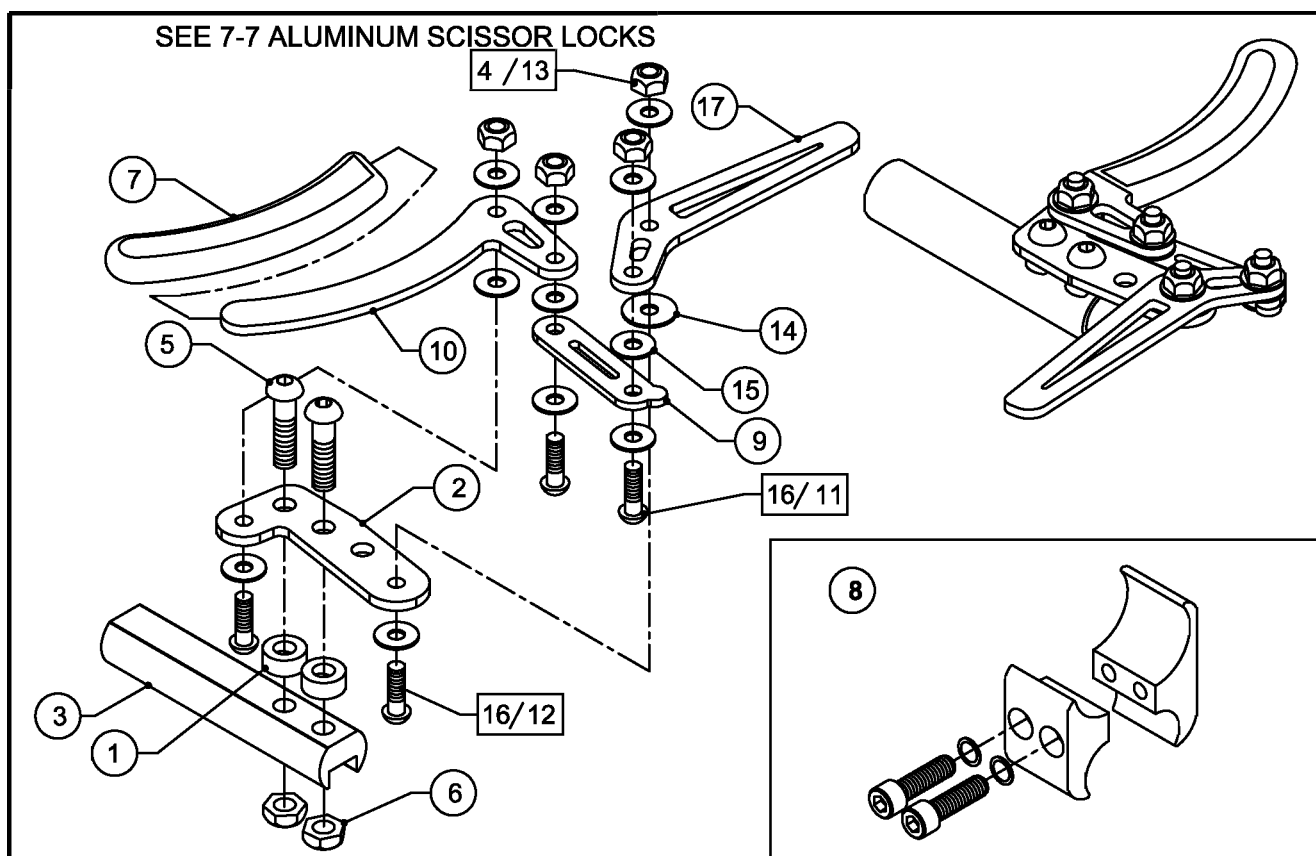

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	HD100016	NYLON STANDOFF .500 O.D. X .257 I.D. X .25 LONG
2	1	CP100018	WHEEL LOCK MOUNT
3	1	CP100180	WHEEL LOCK ADJUSTABLE MOUNT
4	4	HD100019	10-32 FLEX LOCK NUT
5	2	HD100080	1/4-28 X 1" BUTTON HEAD CAP SCREW
6	2	HD100021	1/4- 28 NYLOCK NUT CLEAR ZINC PLATE
7	1	HD100024	WHEEL LOCK FOAM HANDLE
8A	1	CP100048	1" TO 3/4" CLAMP
8B	1	AS102128	3/4" TO 3/4" CLAMP
8C	1	AS102342	1-1/4" TO 3/4" WHEEL LOCK CLAMP ASSEMBLY
9	1	CP100015	WHEEL LOCK LINK
10	1	CP100016	WHEEL LOCK HANDLE
11	1	HD100117	10-32 X 7/8" BUTTON HEAD CAP SCREW (POLY TIRES)
12	1	HD100076	1/4-28 X 7/8" BUTTON HEAD CAP SCREW (POLY TIRES)
13	1	HD100077	1/4-28 TOP LOCK NUT GRADE C (POLY TIRES)
14A	1	HD100017	NYLON WASHER .625 O.D. X .194 I.D. X .031 THICK
14B	3	HD100067	NYLON WASHER .500 O.D. X .265 I.D. X .031 THICK
15A	11	HD100018	NYLON WASHER .505 O.D. X .192 I.D. X .031 THICK
15B	9	HD100018	NYLON WASHER .505 O.D. X .192 I.D. X .031 THICK
16A	4	HD100070	10-32 X 5/8" BUTTON HEAD CAP SCREW
16B	2	HD100070	10-32 X 5/8" BUTTON HEAD CAP SCREW
17A	1	CP100017	WHEEL LOCK STOP STD
17B	1	CP100047	WHEEL LOCK STOP LONG
17C	1	CP100681	WHEEL LOCK STOP STD (POLY TIRE)
17D	1	CP100680	WHEEL LOCK STOP LONG (POLY TIRE)

WHEEL LOCKS (DISCONTINUED)

SCISSOR LOCKS FITS ALL CHAIRS, EXCEPT TiLite™ SX

FOR CHAIRS WITH 1" DIAMETER FRAME TUBES

ITEM	QTY	PART NO.	DESCRIPTION
1-8A,9,10,14A,15A,16A,17A		AS100004	WHEEL LOCK ASSY, RIGHT, STD LNGTH WHL STOP
1-8A,9,10,14A,15A,16A,17A		AS100005	WHEEL LOCK ASSY, LEFT, STD LNGTH WHL STOP
1-8A,9,10,14A,15A,16A,17B		AS100006	WHEEL LOCK ASSY, RIGHT, LONGER LNGTH WHL STOP
1-8A,9,10,14A,15A,16A,17B		AS100007	WHEEL LOCK ASSY, LEFT, LONGER LNGTH WHL STOP
1-8A,9-13,14B,15B,16B,17C		AS100004M	WHEEL LOCK ASSY, RIGHT, STD WHL STOP (POLY TIRES)
1-8A,9-13,14B,15B,16B,17C		AS100005M	WHEEL LOCK ASSY, LEFT, STD WHL STOP (POLY TIRES)
1-8A,9-13,14B,15B,16B,17D		AS100006M	WHEEL LOCK ASSY, RIGHT, LONG WHL STOP (POLY TIRES)
1-8A,9-13,14B,15B,16B,17D		AS100007M	WHEEL LOCK ASSY, LEFT, LONG WHL STOP (POLY TIRES)


FOR CHAIRS WITH 1-1/4" DIAMETER FRAME TUBES

ITEM	QTY	PART NO.	DESCRIPTION
1-8C,9,10,14A,15A,16A,17A		AS102343	1-1/4" STANDARD WHEEL LOCK ASSEMBLY RIGHT
1-8C,9,10,14A,15A,16A,17A		AS102344	1-1/4" STANDARD WHEEL LOCK ASSEMBLY LEFT
1-8C,9,10,14A,15A,16A,17B		AS102345	1-1/4" LONG WHEEL LOCK ASSEMBLY RIGHT
1-8C,9,10,14A,15A,16A,17B		AS102346	1-1/4" LONG WHEEL LOCK ASSEMBLY LEFT
1-8C,9-13,14B,15B,16B,17C		AS102347	1-1/4" STANDARD MAKO WHEEL LOCK ASSEMBLY RIGHT
1-8C,9-13,14B,15B,16B,17C		AS102348	1-1/4" STANDARD MAKO WHEEL LOCK ASSEMBLY LEFT
1-8C,9-13,14B,15B,16B,17D		AS102349	1-1/4" LONG MAKO WHEEL LOCK ASSEMBLY RIGHT
1-8C,9-13,14B,15B,16B,17D		AS102350	1-1/4" LONG MAKO WHEEL LOCK ASSEMBLY LEFT



OFFSET WHEEL LOCK CLAMP


ALL TiLite™ CHAIRS

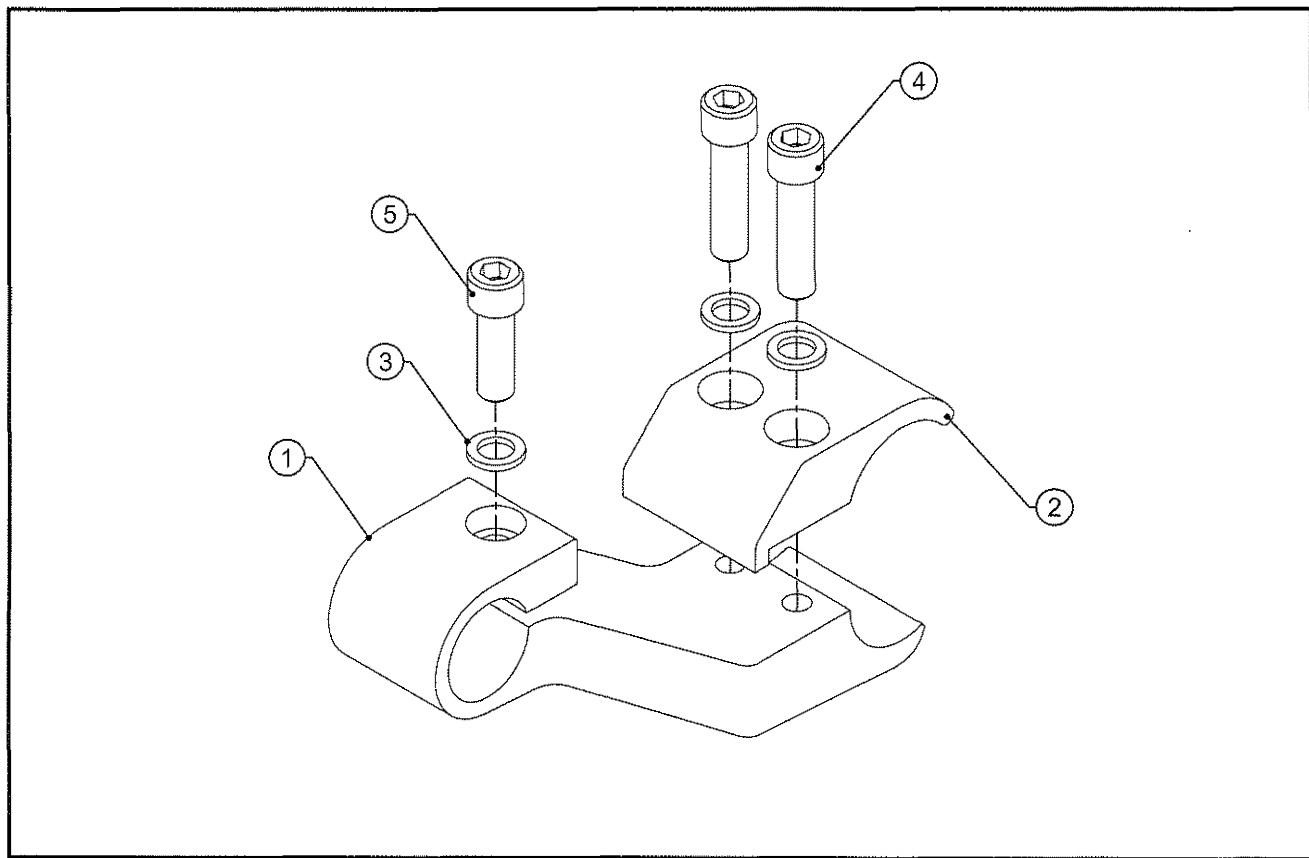

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103415	OFFSET WHEEL LOCK CLAMP MOUNT LEFT
1B	1	CP103432	OFFSET WHEEL LOCK CLAMP MOUNT RIGHT
1C	1	CP103854	1-1/4" OFFSET WHEEL LOCK CLAMP MOUNT, LEFT
1D	1	CP103855	1-1/4" OFFSET WHEEL LOCK CLAMP MOUNT, RIGHT
2A	1	CP103416	OFFSET WHEEL LOCK TOE CLAMP
2B	1	CP103856	1-1/4" OFFSET WHEEL LOCK TOE CLAMP
3	3	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
4	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
5	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
1A,2A-5		AS103681	OFFSET WHEEL LOCK ASSEMBLY, LEFT
1B,2A-5		AS103683	OFFSET WHEEL LOCK ASSEMBLY, RIGHT
1A,1B,2A-5		AS103684	OFFSET WHEEL LOCK ASSEMBLY, PAIR
1C,2B-5		AS104423	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", LEFT
1D,2B-5		AS104424	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", RIGHT
1C,1D,2B-5		AS104425	OFFSET WHEEL LOCK ASSEMBLY 1-1/4", PAIR

EXTENDED WHEEL LOCK CLAMPS


TiLite™ TR, TRC, TRA, YR, YRA, BB, TX, XC, X, SX, Aero X, 2GX,
YG, YGS, EVO, AERO R, TR (SERIES 2), YR (SERIES 2), TWIST, TR (SERIES 3)
AERO T, TRA (SERIES 2)

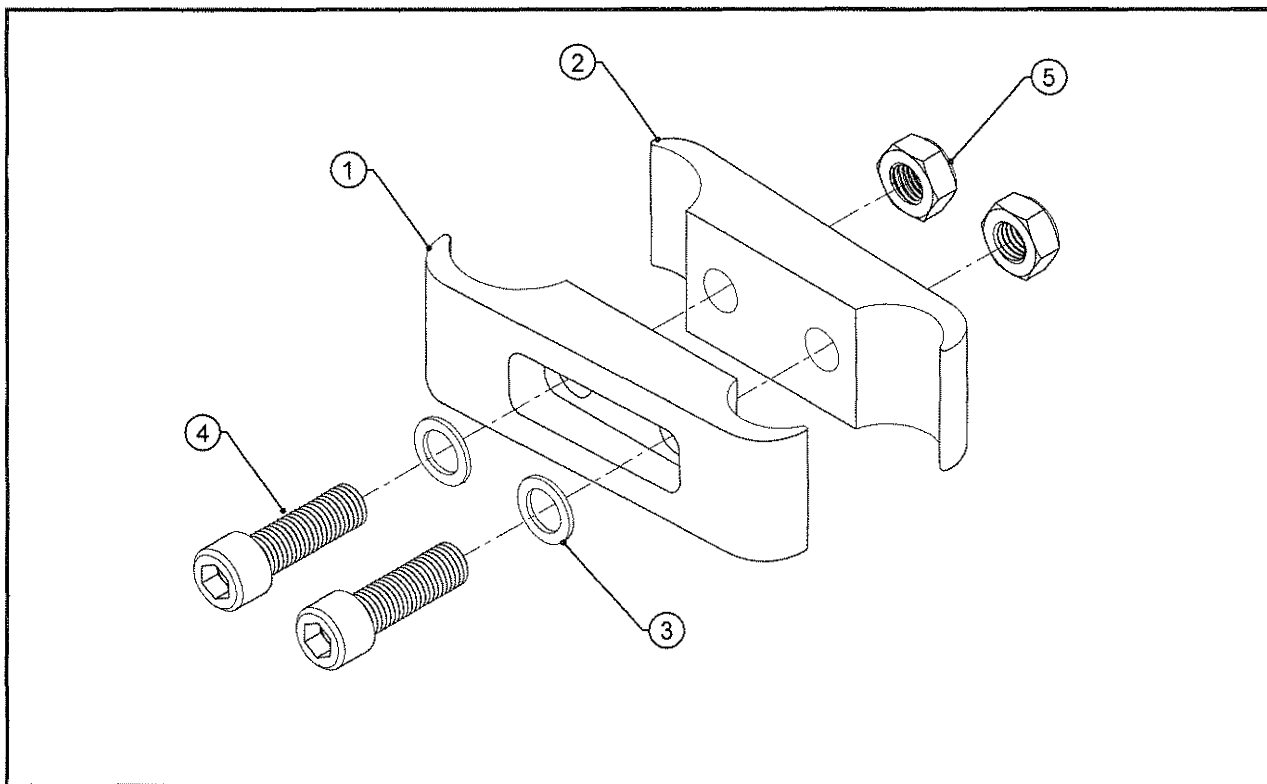

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP102938	EXTENDED WHEEL LOCK CLAMP A FOR Ø1.00
2	1	CP102939	EXTENDED WHEEL LOCK CLAMP B FOR Ø1.00
3	2	HD100432	WASHER Ø.500 X Ø.333 X .04 THK, ZINC PLATE
4	2	HD100433	5/16-24 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC
5	2	HD100434	5/16-24 NYLOCK LOCKNUT, THIN STYLE, BLACK ZINC
1-5		AS103179	EXTENDED WHEEL LOCK CLAMP FOR Ø1.00
1-5		AS103180	EXTENDED WHEEL LOCK CLAMP FOR Ø1.00, PAIR

EXTENDED WHEELLOCK CLAMPS

ALL TiLite™ CHAIRS

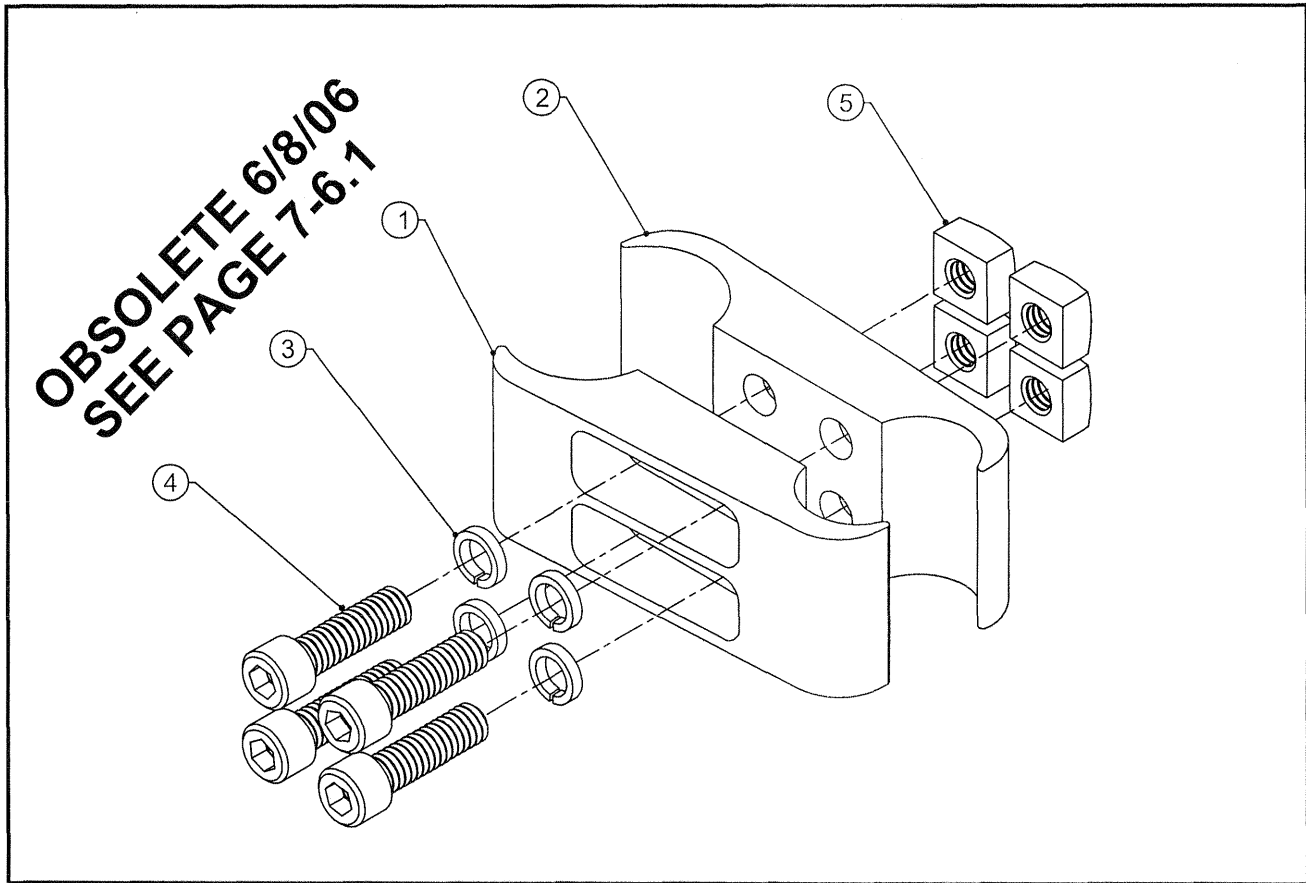

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP102405	1" EXT FOR Ø1.00" WHEELLOCK CLAMP A
1B	1	CP102407	2" EXT FOR Ø1.00" WHEELLOCK CLAMP A
1C	1	CP102409	1" EXT FOR Ø1.25" WHEELLOCK CLAMP A
1D	1	CP102411	2" EXT FOR Ø1.25" WHEELLOCK CLAMP A
2A	1	CP102406	1" EXT FOR Ø1.00" WHEELLOCK CLAMP B
2B	1	CP102408	2" EXT FOR Ø1.00" WHEELLOCK CLAMP B
2C	1	CP102410	1" EXT FOR Ø1.25" WHEELLOCK CLAMP B
2D	1	CP102412	2" EXT FOR Ø1.25" WHEELLOCK CLAMP B
3	4	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
4	4	HD100375	1/4-20 X 7/8" SOCKET HEAD CAP SCREW, BLACK ZINC
5	4	HD100376	1/4-20 SQUARE NUT (7/16" WIDE X 7/32" TALL), BLACK ZINC
1A,2A,3-5		AS102910	1" EXT FOR Ø1.00" WHEELLOCK CLAMP
1B,2B,3-5		AS102911	2" EXT FOR Ø1.00" WHEELLOCK CLAMP
1C,2C,3-5		AS102912	1" EXT FOR Ø1.25" WHEELLOCK CLAMP
1D,2D,3-5		AS102913	2" EXT FOR Ø1.25" WHEELLOCK CLAMP