 For additional information about using this cover, refer to the instructions provided with a compatible ROHO Mattress Overlay.

Supplier: This manual must be given to the user of this product.
Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.
Incident Reporting: In the event of a serious incident or injury, contact Customer Support.
“ Warning” Indicates that not following the specified instructions may lead to hazardous conditions resulting in serious injury.

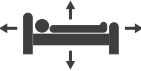
Intended Use: The ROHO Overlay Enclosure Cover is a fluid-resistant textile enclosure, intended to protect a ROHO Mattress Overlay from solid debris or liquids.
Compatibility: Compatible mattress overlays are shown on the cover care label.
In addition to the intended use statement, a clinician should be consulted regarding use of this cover within the individual's overall care regimen and including any need for large-print instructions or other assistive technology.

Cover Placement

 **Warnings:** Periodically, inspect the cover for damage and replace if necessary. • If you are unable to perform any steps, seek assistance. • The mattress overlay surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the overlay will contact unprotected skin. • DO NOT use the handles on the ROHO Overlay Enclosure Cover to lift or slide the individual, as it may result in injury. • DO NOT use the product as a transfer surface.

- Place the cover on the bed mattress, referring to the directional labels to orient the cover.
- Unzip and fold back the top part of the cover.
- Place the mattress overlay sections onto the bottom part of the cover, air cells facing up. Align the mattress overlay sections and snap them together. *Note: If you are using an Insert Pad or Leveling Pad, refer to the attachment instructions provided with the product.*
- Place the top part of the cover over the mattress overlay, but leave the cover at least partially open. You will need to access the inflation valves on one side of the cover; the valves on the other side should all be closed.
- Wrap the straps under the bed mattress and secure the buckles.

 **IMPORTANT!** Follow the inflation and adjustment instructions provided with the mattress overlay.

Other Coverings: If needed, the following items may be placed on top of the mattress overlay and the Overlay Enclosure Cover:

- One flat or contoured sheet, arranged loosely
- One incontinent pad
- One draw sheet, for moving and transferring

 **Warning:** Placing too many linens between the individual and the product will reduce its effectiveness.

Cover Removal:
The individual should transfer to a different surface.

To remove the Overlay Enclosure Cover and the overlay SEPARATELY: Unzip and fold back the top of the cover. Remove the overlay (unsnap and remove sections, if needed). Unbuckle the straps. Remove the cover.

To remove the Overlay Enclosure Cover and the overlay TOGETHER: Do not unzip the cover. The overlay should stay inside the cover. Unbuckle the straps and remove the cover and overlay from the bed mattress.

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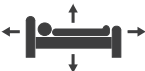
Materiales: Two-way stretch, moisture-resistant fabric with zipper, straps and a carrying handle. Refer to the cover care label for material content and for compliance to flammability requirements.
Cover expected lifetime: 1 year
Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incinerate with other healthcare waste. Incineration must be performed by a qualified, licensed waste management facility.

Cover Care

-  **Warnings:**
- Clean the product regularly and when it becomes soiled. If the product becomes soiled, or between use by different individuals: clean, disinfect, and check for proper functioning.
 - Follow all manufacturer bleach container and germicidal disinfectant safety guidelines.
 - This product does not require sterilization; DO NOT autoclave.


To Clean: Remove the cover from the overlay. Machine wash cold with mild detergent, gentle cycle, or wipe clean with neutral detergent and water. Rinse thoroughly. Tumble dry low or air dry.

To Disinfect: Hand wash, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. Quaternary-type disinfectants may also be used in concentrations recommended by the manufacturer. Rinse thoroughly. Or machine wash in hot water (60°C). Tumble dry low or air dry.

Notes: Repeated washing at high temperatures can accelerate deterioration of the cover and is not recommended). • Quaternary-type disinfectants may also be used in concentrations recommended by the manufacturer.

Note about sanitization: Washing machines certified to NSF Protocol P172 are designed to heat water to 60°C. Recommendation: When washing by hand, use room-temperature water.

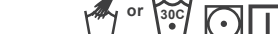Symbols Glossary					
 Model number		 Consult instructions for use.	 Fall risk.	 Machine wash hot, normal, at the temperature shown.	 Hand wash.
 Batch code		 Date of manufacture	 Entrapment risk	 Bleach (1 part bleach: 9 parts water).	 Tumble dry low.
 Fits	 Size	 Manufacturer	 Suffocation risk	 Machine wash cold, gentle.	 Line dry.

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Product information is changed as needed; current product information is available at permobilus.com. • ROHO® is a registered trademark of Permobil. • ROHO, Inc. is a part of Permobil Seating and Positioning.