


How to remove the SpeedControl Dial from a wheelchair frame

Option A: direct attachment

1. Locate the clamp and bolt.
2. Find the clamp holding the SpeedControl Dial to the wheelchair frame. It wraps around the frame tube and is attached with a square head bolt and hex nut.
3. Loosen the clamp.
4. Use a flathead screwdriver or the provided external hex wrench to loosen the hex nut until the clamp starts to open.
5. Remove the hex nut and bolt.
6. Unscrew the hex nut completely and slide the bolt out of the clamp and housing.
7. Unhook the clamp.
8. Gently pull the clamp away from the housing and remove it from the frame tube.
9. Remove the rubber insert (if used) If a rubber insert was used between the clamp and frame, slide it off the tubing.
10. Unclip and disconnect the cable.
11. Disconnect the cable from the SmartDrive system and carefully remove any hook-and-loop strips or tie-straps used to secure it to the chair.

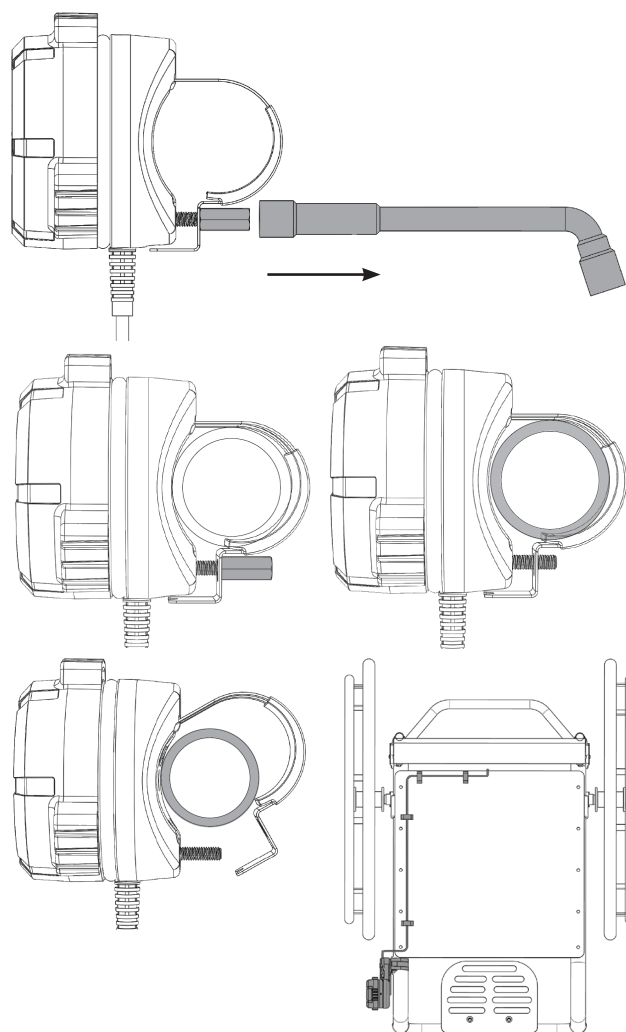

How to remove the SpeedControl Dial from a wheelchair frame

Option B: with extension arm

1. Locate the clamp and arm assembly.
2. Find where the extension arm is attached to the frame using a clamp, bolt, and hex nut.
3. Loosen the hex nut.
4. Use a flathead screwdriver or the provided hex wrench to loosen the hex nut on the bolt.
5. Remove the hex nut and bolt.
6. Unscrew the hex nut completely and slide the bolt out of the arm bracket and clamp.
7. Detach the arm from the clamp.
8. Carefully unhook the top of the arm from the clamp.
9. Remove the clamp and rubber insert (if used).
10. Slide the clamp and any rubber insert off the frame tube.
11. Unclip and disconnect the cable
12. Disconnect the cable from SmartDrive and remove any tie-straps or Velcro securing it to the chair.

