



R-NET
TECHNICAL MANUAL
SK77981/6

I

R-NET TECHNICAL MANUAL PG DRIVES TECHNOLOGY

# © PG Drives Technology 2009

All rights reserved.

This manual is furnished under copyright and may only be used in accordance with the terms laid out by PG Drives Technology.

The information in this manual is furnished for informational use only, is subject to change without notice, and should not be construed as a commitment by PG Drives Technology.

Except as permitted by such terms, no part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means electronic, mechanical, recording, or otherwise - without the prior written permission of PG Drives Technology. +44 (0)1425 271444

# TABLE OF CONTENTS

			5.2	Momentary Screens	21
About	t this manual	0	5.3	Diagnostic Screen	21
ADOU			5.3.1	Identified Module	21
<b>~</b> L	t I Oti		5.3.2	Trip Text	21
Cnap	ter I - Operation		5.3.3	Trip Code	21
1	Introduction	13	5.3.4	Diagnostic Procedure	21
2	General	13	5.4	Locking the Control System	22
2.1	Handling	13	5.4.1	Keypad Locking	22
2.2	Operating Conditions		5.4.2	Key Locking	23
2.3	Cleaning		5.5	Actuator Selection Screen	23
3	Mating Connectors		6	LCD Screen - Color	24
	-		6.1	Top Bar	24
4	Controls	15		Battery Indicator	
4.1	Joystick	16		Focus	
4.2	Buttons	16	6.2	Base Bar	
4.2.	1 On/Off Button	16		Current Profile	
4.2.2	2 Horn Button	16		Motor Temperature	
4.2.3	3 Speed Decrease Button	16		Control System Temperature	
4.2.4	4 Speed Increase Button	17	6.3	Main Screen Area	
4.2.5	5 Mode Button	17		e Screen ::	
4.2.0	6 Profile Button	17		Profile Name	
4.2.7	7 Hazard Warning Button and LED	17		Clock	
4.2.8	3 Lights Button and LED	17		Speed Display	
4.2.9	Left Indicator Button and LED	17		Maximum Speed Indicator	
4.2.	10 Right Indicator Button and LED	17		Digital Speed Display	
	11 External On/Off Switch Jack			Latched	
4.2.	12 External Profile Switch Jack	18		Inhibit	
4.3	LCD Screen	18		de Screens ::	
4.4	Charger Socket	18		Actuator Mode	
5	LCD Screen - Monochrome			Bluetooth Mode	
5.1	Screen Symbols		6.4	Message Window	
5.1.				Restart	
	2 Speed Indicator			Timer	
	3 Current Profile			Sleep	
	4 In Focus			Cross & Tick	
	5 Speed Limit			E-stop	
				Joystick Displaced	
	5 Latched				
	Residit			Control System Locked	
				Diagnostic Screen	
	9 Motor Temperature		7	Getting Ready to Drive	29
	10 Control System Temperature		8	Settings Menu	30
	12 E-Stop		8.1	Set Time	30
	·		8.2	Display Time	30
٥.١.	13 Bluetooth	∠∪	8.3	Distance	

8.4	Backlight	31	4.2.3	Position	
8.5	Background	31	4.3	ISM Mounting	
8.6	IR Setup	31	4.3.1	General	
8.7	Exit	31	4.3.2	Orientation	
9	Tips for Using Your Control System	32	4.3.3	Position	
9.1	Driving - General	32	4.4	Cables	
9.2	Driving Technique		5	Joystick Module Wiring	
9.3	Slow or sluggish movement	32	5.1	Charger Connector	
10	Precautions for Use	32	6	Power Module & ISM Wiring	
10.1	Hazards	32	6.1	General	
11	Safety Checks	34	6.1.1	Power Module Wiring	
11.1	Daily Checks	34	6.1.2	ISM Wiring	
11.2	Weekly Checks		6.1.3	Crimping	
11.3	Servicing		6.2	Wire Gauge and Types	
12	Battery Gauge		6.3	Battery Connections	
			6.4	Motor Connections	
12.1	How to Read a Battery Gauge		6.5	Inhibit Connections	
13	Battery Charging		6.6	Lighting Connections	
14	Programming	37	6.7	Indicator Connections	
15	Joystick Knobs	37	6.8	Actuator Connections	
	Servicing		6.9	On-Board Charger Connections	
16	•		7	Batteries	
17	Warranty	38	7.1	Battery Charging	
S			8	Drive Motors	
cnapt	er 2 - INSTALLATION		9	Solenoid Brakes	
1	Documentation		10	Programmer Connection	
1.1	R-net Operation	41			
1.2	Program Settings	41	10.1	OBP Programming	
1.3	Soft-Stop	41		1 Navigating the OBP Screens	
1.4	Other Information	42		PC Programming	
2	Immobilizing the Wheelchair	42		DTT Programming	
2.1	Prevention of Unauthorized Use	42	10.3	3	
2.2	Charger Interlock			Production Tests	
2.3	Power Down		11.1	Mounting	
3	Connections	43	11.2	Cables and Connectors	
3.1	Control Configurations	43	11.3	Joystick and Gaiter	
3.1.1			11.4	Preset Settings	
	Joystick & ISM Configuration		11.5	Operational Test Test Drive	
4	Mounting		11.6		
			11.7	Soft-Stop TestLights, Indicators and Hazard Lamps Test	
4.1	Joystick Module Mounting General		11.8	Actuator Test	
4.1.1					
4.1.2 4.2				) Inhibit Input Test	
4.2.1	Power Module Mounting		12	Electromagnetic Compatibility (E.M.C.)	
	General	44 45	12.1	Emissions	

12.1	.1 Motor Suppression	58	6.13	Maximum Reverse Deceleration	77
12.1	.2 Cables	58	6.14	Minimum Reverse Deceleration	77
12.2	Immunity	58	6.15	Maximum Turn Acceleration	78
12.3	Electro-Static Discharge (E.S.D.)	58	6.16	Minimum Turn Acceleration	78
13	Battery Gauge	59	6.17	Maximum Turn Deceleration	78
			6.18	Minimum Turn Deceleration	78
Chapt	ter 3 – Programming	6I	6.19	Power	78
1	Introduction		6.20	Torque	79
-			6.21	Tremor Damping	
2	On-Board Programming	63	6.22	Fast Brake Rate	
2.1	How to enter OBP Mode	63	7	Controls	79
2.2	Navigating the OBP Screens	64	Contr	rols - Global	79
2.3	Making an Adjustment	64	7.1	Momentary Screens Enabled	79
3	PC Programmer - OEM Parameter List	65	7.2	Change Profile while Driving	79
4	Profile Management	69	7.3	Change Speed while Driving	79
·· Dro	ofiles And Modes Explained ::		7.4	Speed Adjust	79
	file Management:		7.5	Profile Button	80
	de Management :		7.6	Actuator Endstop Beep	80
	ecting Profile and Modes with Just One Button :		7.7	Sounder Volume	80
4.1	Profile Name		7.8	Lock Function Enabled	80
	Profile Enable - (PrfEn)		7.9	Reverse Driving Alarm	80
4.2	• • •		7.10	Emergency Stop Switch	80
4.3	Mode Enable (Modes)		7.11	OBP Keycode Entry	
-	but Devices Explained ::		7.12	Power-up Mode	
4.4	Input Device Type - (ID Type)			rols - Joystick	
4.5	Input Device Subtype		7.13	Joystick Forward Throw	
4.6	Seat Reversal Profile		7.14	Joystick Reverse Throw	
4.7	Allow Grab		7.15	Joystick Left Throw	
5	Configuration	/4	7.16	Joystick Right Throw	
	onfiguration Explained ::		7.17	Joystick Deadband	
: Use	r Displays :	74	7.18	Invert Fwd Rev JS Axis , Invert Left Right JS Axis &	
5.1	Mode Name	74		Swap Joystick Axis	83
5.2	Input		Contr	rols – Profiled	83
5.3	Output	75	7.19	Change Mode while Driving	83
6	Speeds	76	7.20	Sleep Timer	83
6.1	Maximum Forward Speed	76	:: Stai	ndby Operation Explained ::	84
6.2	Minimum Forward Speed	76	7.21	Standby Time	84
6.3	Maximum Reverse Speed	76	7.22	Switch to Standby	84
6.4	Minimum Reverse Speed		7.23	Mode Selection in Standby	84
6.5	Maximum Turning Speed		7.24	Standby in Modes	84
6.6	Minimum Turning Speed		7.25	Standby Forward	85
6.7	Maximum Forward Acceleration		7.26	Standby Reverse	85
6.8	Minimum Forward Acceleration		7.27	Standby Left	86
6.9	Maximum Forward Deceleration		7.28	Standby Right	86
6.10	Minimum Forward Deceleration		7.29	Remote Selection	86
6.11	Maximum Reverse Acceleration)		7.30	Background	86
6.12	· ·		8	Latched	87

8.1	Latched Drive	. 87	12.4	Invert M1 Direction	98
8.2	Latched Actuators	. 87	12.5	Invert M2 Direction	98
8.3	Latched Timeout	. 88	12.6	Motor Swap	98
8.4	Latched Timeout Beep	. 88	12.7	Steer Correct	99
9	Seating	88	12.8	Display Speed - (Display Speed)	99
9.0.1	Seating Module Enabled	88	12.9	Maximum Rated Speed	99
	uator Axes Explained ::		12.10	Maximum Displayed Speed (Max Display Speed	l) 99
	gent Seating Module - Global		Moto	r - Profiled	100
9.1	Actuator Mode Entry Axis		12.11	Compensation Factor	100
	gent Seating Module - Axes Setup		13	Inhibits	101
9.2	Axis Channels, Axis Display, Invert Axis Direction		:: Inhi	bits Explained ::	101
9.3	Axis Name			eed Inhibit Example ::	
	gent Seating Module - Channels		-	uator Inhibit Examples ::	
9.4	Acceleration			ts - Inputs - PM	
9.5	Deceleration			Lower Level Threshold	
9.6	Up Speed	. 90		Middle Level Threshold	
9.7	Down Speed		13.3	Upper Level Threshold	
9.8	Up Current Limit			Debounce Time	
9.9	Down Current Limit			ts - Inputs – ISM	
9.10	Endstop Autodetect		13.5	Lower Level Threshold	
9.11	Actuator Endstop Timeout		13.6	Middle Level Threshold	
10	Battery		13.7	Upper Level Threshold	
	Low Battery Alarm		13.8	Debounce Time	
10.1	Low Battery Flash Level			ts - Drive Inhibits	
10.2	•		13.9	Assign	105
10.3	Cable Resistance			) Speed Limit in Band 0	
10.4	Calibration Factor			Speed Limit in Band 1	
10.5	Low Voltage Cut-out, Low Voltage Time			? Speed Limit in Band 2	
11	General	. 93		S Speed Limit in Band 3	
11.1	Brake Disconnected Alarm			· ! Operation	
11.2	Brake Fault Detect			5 Inhibit Alarm	
11.3	Brake Voltage		Inhibi	ts - Actuator Inhibits	107
11.4	•			Channel x - Up: Assign (x = 1 to 6)	
11.5	Front Wheel Drive	. 93		Channel x – Up: Inhibit in Bands (x = 1 to 6)	
11.6	Seat Reversal	. 93		Channel x – Up: Alarm	
11.7	Charger Inhibit 3 Enable	. 94		Channel x – Down: Assign (x = 1 to 6)	
11.8	Charger Inhibit 3 Latching	. 94		Channel x – Down: Inhibit in Bands (x = 1 to 6)	
11.9	Charger Inhibit 1 Latching	. 95		Channel x – Down: Alarm	
11.10	) Charger Inhibit 1 Alarm	. 95	14	Lights	
11.11	Charger Inhibit 3 Alarm	. 95		_	
12	Motor	96	14.1	Lamp Voltage	
Moto	r - Global	. 96	14.2 14.3	Swap Indicators Seat Reversal	
12.1	Maximum Current Limit, Boost Drive Current, Boost			·	
	Drive Time	. 96	14.4	Brake Lights or Horn	
12.2	Current Foldback Threshold, Current Foldback Time,		15	OEM Factory	110
,	Current Foldback Level				
12.3	Compensation	. 9/			

Chapt	er 4 – ELPM 90, JSM-LED & SM	III	7.2	Actuator Mode Entry Axis - (Act. Entry Axis)	125
1	Introduction	. 113	Seatir	ng Module - Axes Setup	126
_	Introduction		7.3	Axis Enabled	126
1			7.4	Up Speed - (UpSpd)	
2	Controls – JSM-LED and JSM-LED-L	. 114	7.5	Down Speed - (DnSpd)	
2.1	Buttons	115	7.6.1	Axis End Force	
2.1.1	Battery Gauge	115	7.7	Actuator Current Limit Timeout	
2.1.2	Maximum Speed / Profile Indicator	115	7.6	Axis Display	
2.1.2	.1 Maximum Speed Indicator	115		ng Module - Axes Inhibits	
2.1.2	.2 Profile Indicator	116	7.7	Axis x – Up Assign (x = 1 to 12)	
2.1.3	Speed / Profile Decrease Button	116	7.78	Axis $x - Up$ Inhibit in Bands ( $x = 1 \text{ to } 6$ )	
2.1.4	Speed / Profile Increase Button	116	7.9	Axis $x$ – Down Assign ( $x$ = 1 to 12)	
2.1.5			7.10	Axis $x - Down$ : Inhibit in Bands ( $x = 1 \text{ to 6}$ )	129
2.2	Actuator Indicator	116			
3	Control System Status indication	. 117	Chapt	er 5 – Encoder Module	
3.1	Battery Gauge is Steady	117	1	Introduction	133
3.2	Battery Gauge Flashes Slowly	117	2	Installation	133
3.3	Battery Gauge Steps Up	117	3	Mounting	134
3.4	Battery Gauge Flashes Rapidly (even with the joystick released)	117	3.1	Orientation	
3.5	Speed Indicator Ripples Outwards		3.1	Position	
3.6	Speed Indicator LEDs 2 & 4 Flash		3.3	Connections	
3.7	Self-Help Guide		4	Programming	
3.8	Slow or Sluggish Movement			-	
3.9	Maximum Speed / Profile Indicator is Steady		4.1	Encoder Module Fitted	
	1 Speed Indication		4.2	Encoder Error Speed Limit	
	2 Profile Indication		4.3	Invert M1	
3.11	Maximum Speed / Profile Indicator Ripples Up an		4.4	Invert M2	
0,,,	Down		4.5	Encoder Resolution	
3.12	Maximum Speed / Profile Indicator Flashes	119	4.6	Motor Rated Speed	
4	Getting Ready to Drive	. 119	4.7	Compensation	130
5	Mounting and Installation	. 120	Chapt	er 6 – Diagnostics	רכו
6	Power Module & SM Wiring	. 121	_	_	
6.1	General		1	Introduction	
6.1.1			1.1	Diagnostic Procedure	
	Seating Module Wiring		2	Diagnostic Screens	139
	Crimping		2.1	Current Diagnostic Screen	139
6.2	Wire Gauge and Types		2.1.1	Identified Module	140
6.3	Battery and Motor Connections		2.1.2	Trip Text	140
6.4	EL PM to Seating Module Connections		2.1.3	Trip Code	140
6.5	Inhibit Connections		2.1.4	Example	140
6.6	Actuator Connections.		2.2	System Log Screen	140
7	Programming		2.3	Connections	141
			3	Diagnostic Text Definitions	142
	ruator Axes Explained ::		3.1	Center Joystick	142
7.1	Seating Module Enabled		3.2	Low Battery	
seatii	ng Module - Global	125		,	-

3.3	High Battery Voltage	143	2.9	Warranty	157
3.4	Brake Error	143	2.10	Program Settings	157
3.5	Motor Error	143	2.11	Charger Interlock	157
3.6	Inhibit Active	143	2.12	Power Module Wiring - General	157
3.7	Joystick Calibration Error	144	2.13	Crimping	158
3.8	Brake Lamp Short	144	2.14	Wire Gauge and Types	158
3.9	Lamp Short	144	2.15	Battery Connections	
3.10	Indicator Lamp Short	144	2.16	Battery Charging	158
3.11	Indicator Lamp Failed	144	2.17	Drive Motors	158
3.12	Over-current	144	2.18	Solenoid Brakes	159
3.13	Overtemp (acts)	145	2.19	Introduction	159
3.14	Overtemp (lamps)		2.20	How to Enter OBP Mode	159
3.15	DIME Error		2.21	Tremor Damping	
3.16	Memory Error	145	2.22	Emergency Stop Switch	
3.17	PM Memory Error		2.23	OBP Keycode Entry	
3.18	Bad Cable		2.24	Latched Drive	
3.19	Bad Settings	146	2.25	Latched Timeout	
3.20	Module Error		2.26	Actuator Acceleration	160
3.21	System Error		2.27	Actuator Deceleration	160
3.22	SID Disconnected		2.28	Brake Fault Detect	
3.23	Switch Detached		2.29	Brake Voltage	
3.24	Gone to Sleep		2.30	Soft Stop Rate	
3.25	Charging		2.31	Maximum Current Limit, Boost Drive Currer	
3.26	Encoder Error			Drive Time	
4	Basic Tests		2.32	Compensation	161
	General Inspection		2.33	Compensation Factor	161
4.1	·		2.35	Battery Gauge	161
4.2 4.3	Brake Test  Drive Test		2.36	Getting Ready to Drive	161
4.3 4.4	Gradient Test		2.37	General	162
4.4 4.5	Lights, Indicators and Hazard Lamps Test		2.38	Introduction	162
	0 '		2.39	PM Memory Error	162
4.6	Actuator Test		2.40	Basic Tests	162
4.7	Inhibit Input Test		2.41	Servicing of Defective Units	163
5	Servicing of Defective Units	151			
<b>-</b>	<b>-</b> <i>-</i>		Chapt	er 8 – Specifications	165
Cnapt	er 7 – Warning Summary	153	1	Electrical Specifications	167
1	Introduction	155		·	
2	Warnings	155			
2.1	Charger Socket	155			
2.2	Driving Technique	155			
2.3	Hazards	155			
2.4	Battery Charging	156			
2.5	Battery Charging	156			
2.6	Programming	156			
2.7	Joystick Knobs	156			
2.8	Servicing	156			

PG Drives Technology R-net Technical Manual

# About this manual

The Technical Manual gives an introduction to the R-net Control System.

Throughout the manual icons are used to draw the reader's attention.

The icons used are:



Note - A general point for best practice.



Caution - A point of safety which if ignored could result in damage to the Control System or the vehicle.



Warning - A point of safety which if ignored could cause injury to the individual.

PG Drives Technology accepts no liability for any losses of any kind if the points are not followed.

R-NET TECHNICAL MANUAL PG DRIVES TECHNOLOGY

IO SK7798I/6



# **CHAPTER I - OPERATION**

SK77981/6 II

12 SK77981/6

# I Introduction

The relevant contents of this chapter should be included in the wheelchair's operating guide. Further copies are available from PGDT in either written or disk (Word) format. Copies should not be made without the express permission of PG Drives Technology.


The operation of the R-net varies dependent on programming. This chapter covers all types of operation. It is the responsibility of the wheelchair manufacturer to ensure that only the relevant sections of this chapter are included in the wheelchair's operating manual.

The operation of the R-net wheelchair control system is simple and easy to understand. The control system incorporates state-of-the-art electronics, the result of many years of research, to provide you with ease of use and a very high level of safety. In common with other electronic equipment, correct handling and operation of the unit will ensure maximum reliability.

Please read this chapter carefully - it will help you to keep your wheelchair reliable and safe.

# 2 General

An R-net control system comprises a minimum of two modules - Joystick Module and Power Module. Because of the modular design, the depth of the control system can be greatly increased. The following diagram shows the basic set-up.

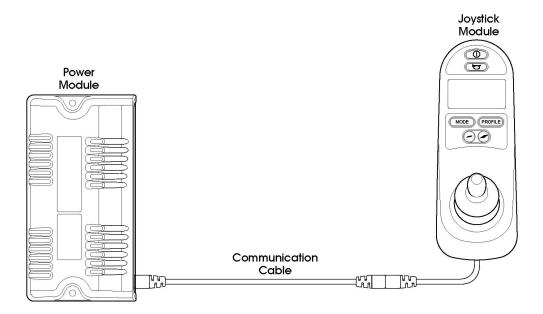

# 2.I Handling

Avoid knocking your control system and especially the joystick. Be careful not to strike obstacles with the control system or joystick when you drive. Never drop the control system.

When transporting your wheelchair, make sure that the control system is well protected. Avoid damage to cables.

#### 2.2 Operating Conditions

Your control system uses industrial-grade components throughout, ensuring reliable operation in a wide range of conditions. However, you will improve the reliability of the control system if you keep exposure to extreme conditions to a minimum.

SK7798I/6 I3

Do not expose your control system or its components to damp for prolonged periods. If the control system becomes contaminated with food or drink clean it off as soon as possible.

#### 2.3 Cleaning

Clean the control system and the joystick with a cloth dampened with diluted detergent. Be careful when cleaning the joystick and screen.


Never use abrasive or spirit-based cleaners.

# 3 Mating Connectors

To connect the Communication Cables:

 Holding the connector housing, firmly push the connector into its mate until you can no longer see the yellow plastic.

The connectors are secured using a friction system.

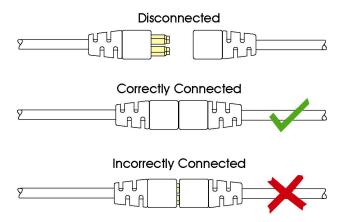

To disconnect the Communication Cables:

Holding the connector housing firmly, pull the connectors apart.



Do not hold or pull on the cable. Always grip the connector when connecting and disconnecting.





When the control system is first switched on after a connection, or system component change the Timer will be displayed whilst the system checks itself and then the Re-start icon will be displayed. Switch the control system off and on again to operate.


I4 SK7798I/6

# 4 Controls

The R-net control system has two versions of Joystick Module – with and without lighting control. Most of the controls are common to both however, the lighting buttons are only included on the Joystick Module with lighting control. Each of the controls is explained within this section.

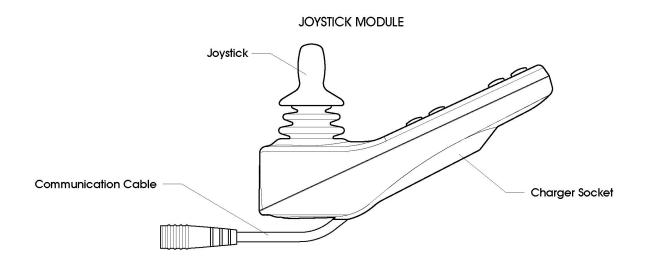

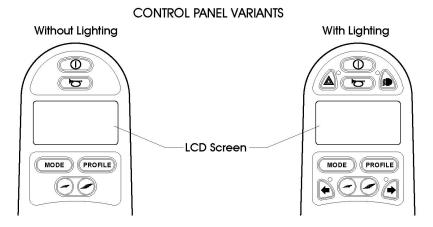

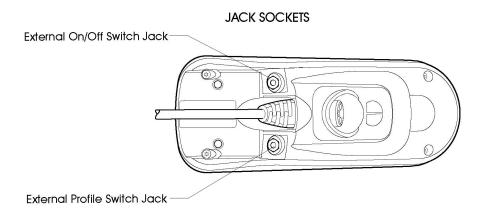


SK7798I/6 I5

### 4.I Joystick

The primary function of the joystick is to control the speed and direction of the wheelchair. The further you push the joystick from the center position the faster the wheelchair will move. When you release the joystick the brakes are automatically applied.

If the wheelchair is fitted with actuators, the joystick can also be used to move and select actuators, refer to section 5.5 for more details.

#### 4.2 Buttons

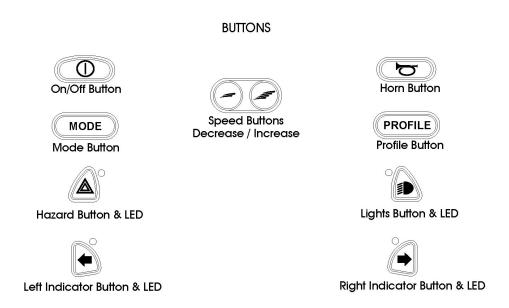

# 4.2.I On/Off Button

The On/Off button applies power to the control system electronics, which in turn supply power to the wheelchair's motors. Do not use the On/Off button to stop the wheelchair unless there is an emergency. (If you do, you may shorten the life of the wheelchair drive components).

### 4.2.2 Horn Button

The Horn will sound while this button is depressed.

### 4.2.3 Speed Decrease Button

This button decreases the maximum speed setting.

Depending on the way the control system has been programmed a momentary screen may be displayed when the button is pressed.

Refer to section 5 for details of the momentary screen.

Refer to Chapter 3 - Programming for details.

I6 SK7798I/6

#### 4.2.4 Speed Increase Button

This button increases the maximum speed setting.

Depending on the way the control system has been programmed a momentary screen may be displayed when the button is pressed.

Refer to section 5 for details of the momentary screen.

Refer to Chapter 3 - Programming for details.

#### 4.2.5 Mod∈ Button

The Mode button allows the user to navigate through the available operating Modes for the control system. The available modes are dependent on programming and the range of auxiliary output devices connected to the control system.

Refer to Chapter 3 - Programming for details.

#### 4.2.6 Profile Button

The Profile button allows the user to navigate through the available Profiles for the control system. The number of available Profiles is dependent on how the control system is programmed.

Depending on the way the control system has been programmed a momentary screen may be displayed when the button is pressed.

Refer to section 5 for details of the momentary screen.

Refer to Chapter 3 - Programming for details.

# 4.2.7 Hazard Warning Button and LED

This button activates and de-activates the wheelchair's hazard lights. Depress the button to turn the hazards on and depress the button again to turn them off.

When activated the hazard LED and the indicator LEDs will flash in sync with the wheelchair's indicators.

### 4.2.8 Lights Button and LED

This button activates and de-activates the wheelchair's lights. Depress the button to turn the lights on and depress the button again to turn them off.

When activated the lights LED will illuminate.

# 4.2.9 Left Indicator Button and LED

This button activates and de-activates the wheelchair's left indicator. Depress the button to turn the indicator on and depress the button again to turn it off.

When activated the left indicator LED will flash in sync with the wheelchair's indicator(s).

### 4.2.10 Right Indicator Button and LED

This button activates and de-activates the wheelchair's right indicator. Depress the button to turn the indicator on and depress the button again to turn it off.

When activated the right indicator LED will flash in sync with the wheelchair's indicator(s).

# 4.2.II External On/Off Switch Jack

This allows the user to turn the control system on and off using an external device, such as a buddy button.

SK7798I/6 I7

#### 4.2.12 External Profile Switch Jack

This allows the user to select Profiles using an external device, such as a buddy button.

To change the Profile whilst driving simply press the button.

If the control system is set to latched drive or actuator control operation, then the polarity of the jack input is reversed to effect a fail safe system; meaning this input will provide an External Profile Switch function and an Emergency Stop Switch function.



The Joystick Module is supplied with rubber bungs that must be inserted into the Jack Socket when no external device is connected.

#### 4.3 LCD Screen

The status of the control system can be understood by observing the LCD screen. The control system is on when the screen is backlit.

Refer to section 5 for details on screen symbols.

### 4.4 Charger Socket

This socket should only be used for charging or locking the wheelchair. Do not connect any type of programming cable into this socket.

Refer to section 13 for more details on charging.

This socket should not be used as a power supply for any other electrical device. Connection of other electrical devices may damage the control system or affect the E.M.C. performance of the wheelchair.




The control system's warranty will be voided if any device other than a battery charger supplied, with the wheelchair, or the lock key is connected into this socket.

# 5 LCD Screen - Monochrome

The status of the control system can be understood by observing the LCD screen.

### 5.I Screen Symbols

The Drive screen for the R-net has common components, which will always appear, and components that will only appear under certain conditions. Below is a view of a typical Drive screen in Profile 1.

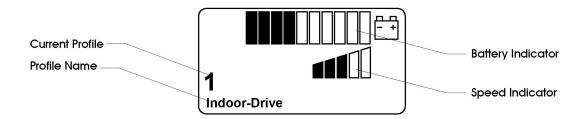

I8 SK7798I/6

#### 5.I.I Battery Indicator

This displays the charge available in the battery and can be used to alert the user to the status of the battery.



#### Steady

This indicates that all is well.

### Flashing Slowly


The control system is functioning correctly, but you should charge the battery as soon as possible.

### Stepping Up

The wheelchair batteries are being charged. You will not be able to drive the wheelchair until the charger is disconnected and you have switched the control system off and on again.

Refer to section 12 for a description of how to read the Battery Gauge.

# 5.I.2 Speed Indicator

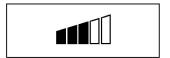

This displays the current speed setting.

The speed setting is adjusted using the Speed Buttons.

### 5.1.3 Current Profile



The Profile Number describes which Profile the control system is currently operating in.

The Profile Text is the name or description of the Profile the control system is currently operating in.

#### 5.I.4 In Focus



When the control system contains more than one method of direct control, such as a secondary Joystick Module or a Dual Attendant Module, then the Module that has control of the wheelchair will display the In Focus symbol.

# 5.1.5 Speed Limit



If the speed of the wheelchair is being limited, for example by a raised seat, then this symbol will be displayed.

If the wheelchair is being inhibited from driving, then the symbol will flash.

# 5.I.6 Latched



When the control system is operating in a latched condition this symbol will be displayed.

SK7798I/6 I9

#### 5.1.7 Restart



When the control system requires a reboot; for example, after a module re-configuration, this symbol will be flashed.

5.I.8 Fault



The control system can detect a wide variety of errors. When the system has detected an error that is not severe enough to cause the system to trip, then this symbol will be displayed.

### 5.I.9 Motor Temperature



This symbol is displayed when the control system has intentionally reduced the power to the motors, in order to protect them against heat damage.

### 5.I.IO Control System Temperature



This symbol is displayed when the control system has intentionally reduced its own power, in order to protect itself against heat damage.

5.I.II Timer



This symbol is displayed when the control system is changing between different states. An example would be entering into Programming Mode. The symbol is animated to show the sands falling.


## 5.1.12 E-Stop



If the control system is programmed for latched drive or actuator operation, then it is normal for an Emergency Stop Switch to be connected into the External Profile Switch Jack.

If the Emergency Stop Switch is operated or disconnected, this symbol will be displayed.


## 5.I.I3 Bluetooth

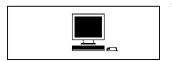

When Bluetooth Mode is entered the screen will display the following icon.

### 5.2 Momentary Screens

If the momentary screens are programmed to be displayed then pressing the Speed or Profile Buttons will display screens such as below

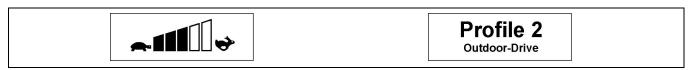


Speed Momentary Screen

Profile Momentary Screen

# 5.3 Diagnostic Screen

When the control system safety circuits have operated and the control system has been prevented from moving the wheelchair a diagnostics screen will be displayed.

This indicates a system trip, i.e. the R-net has detected a problem somewhere in the wheelchair's electrical system.

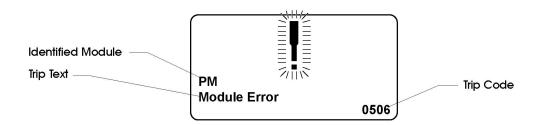

If the error is in a non-active module, for example in the ISM but with a drive Profile is selected, then drive will still be possible, however, the diagnostic screen will appear intermittently.

#### 5.3.1 Identified Module

This identifies which module of the control system has registered the problem, such as:

PM Power Module

JSM Joystick Module

ISM Intelligent Seating/lighting Module

# 5.3.2 Trip Text

The Trip Text gives a brief description of the trip type.

### 5.3.3 Trip Code

The 4 digit code displayed gives the exact trip that has been recorded.

# 5.3.4 Diagnostic Procedure


Please follow this procedure:

- Read and note the Trip Text displayed, the identified Module and the Trip Code.
- Switch off the control system.

SK7798I/6 2I

- Make sure that all connectors on the listed Module and the wheelchair are mated securely.
- Check the condition of the battery.
- Note the Trip Text description, and take the required action.
- Switch on the control system again and try to drive the wheelchair. If the safety circuits operate again, switch off
  and do not try to use the wheelchair. Contact your service agent.

#### Example:

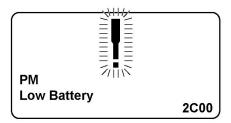

Identified Module Power Module.

Trip Text Low Battery
Trip Code 2C00

This means the battery needs charging or there is a bad connection to the battery.

Check the connections to the battery. If the connections are good, try charging the battery.

### 5.4 Locking the Control System

The Control System can be locked in one of two ways. Either using a button sequence on the keypad or with a physical Key. How the Control System is locked depends on how the wheelchair manufacturer has programmed the system.

# 5.4.1 Keypad Locking

To lock the wheelchair using the keypad;

- While the control system is switched on, depress and hold the On/Off button.
- After 1 second the control system will beep. Now release the On/Off button
- Deflect the joystick forwards until the control system beeps.
- Deflect the joystick in reverse until the control system beeps.
- Release the joystick, there will be a long beep.
- The wheelchair is now locked.

The following screen will be displayed, the next time the Control System is switched on.



If an LED Joystick Module is fitted the Speed Indicator LEDs will ripple from left to right. Refer to Chapter 4.

To unlock the wheelchair:


- If the control system has switched off, press the On/Off button.
- Deflect the joystick forwards until the control system beeps.
- Deflect the joystick in reverse until the control system beeps.
- Release the joystick, there will be a long beep.
- The wheelchair is now unlocked.

#### 5.4.2 Key Locking

To lock the wheelchair with a key;

- With the Control System switched on, insert and remove a PGDT supplied key into the Charger Socket on the Joystick Module. A short beep will be heard.
- The wheelchair is now locked.

The following screen will be displayed, the next time the Control System is switched on.

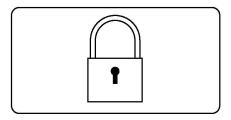


To unlock the wheelchair;

- If the control system has switched off, press the On/Off button.
- Insert and remove a PGDT supplied key into the Charger Socket on the Joystick Module. A short beep will be heard.
- The wheelchair is now unlocked.

### 5.5 Actuator Selection Screen

To adjust the seat position the actuator screen must be visible.

Depress the Mode Button to scroll through the Mode screens until you reach the actuator screen, displayed below.

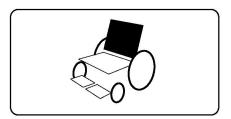


Actuator adjustment is achieved as follows.

• Move the Joystick sideways to select the desired axis.

(This is indicated by the section of the wheelchair that is highlighted)

Move the joystick forwards and backwards to move the actuator.

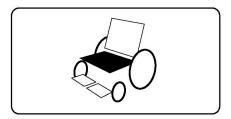


• Repeat these steps for each actuator that requires adjustment.

To drive again depress the Mode button until the Drive screen is reached or, in the case of the LED joystick module, until the Speed Indicator returns to its normal state.

# 6 LCD Screen - Color

This section covers those Joystick Modules that are fitted with a color LCD screen.

The color LCD screen is split into 3 areas of information. The Top Bar, the Base Bar and the Main Screen Area.

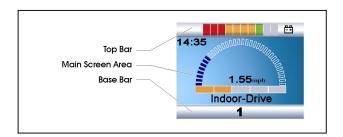

Each area is covered separately within this section.

# 6.I Top Bar

# 6.I.I Battery Indicator



This displays the charge available in the battery and can be used to alert the user to the status of the battery.

**Steady:** This indicates that all is well.

**Flashing Slowly:** The control system is functioning correctly, but you should charge the battery as soon as possible.

**Stepping Up:** The wheelchair batteries are being charged. You will not be able to drive the wheelchair until the charger is disconnected and you have switched the control system off and on again.

Refer to section 12 for a description of how to read the Battery Gauge.

#### 6.1.2 Focus



When the control system contains more than one method of direct control, such as a secondary Joystick Module or a Dual Attendant Module, then the Module that has control of the wheelchair will display the In Focus symbol.

#### 6.2 Base Bar

### 6.2.I Current Profile



The currently selected Profile is shown in numeric form.

### 6.2.2 Motor Temperature



This symbol is displayed when the control system has intentionally reduced the power to the motors, in order to protect them against heat damage.

### 6.2.3 Control System Temperature



This symbol is displayed when the control system has intentionally reduced its own power, in order to protect itself against heat damage.

# 6.3 Main Screen Area

### :: Drive Screen ::

### 6.3.I Profile Name



This is a text string that displays the name of the currently selected Profile. The name is programmable. Refer to the programming section for details.

# 6.3.2 Clock

14:35


This displays the current time in a numeric format.

The clock is user adjustable. Adjustable options are:

- Visibility, whether the clock is displayed on screen.
- The display format, 12 or 24 hour.
- The time, the user can adjust the time.

These adjustments are made within the Settings Menu. Refer to section 8 for details.

### 6.3.3 Speed Display

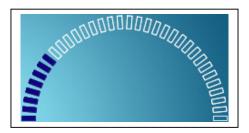

This gives a proportional display of the wheelchairs speed. The Arc begins at 0% and has a programmable maximum. The programmable parameter is Max Displayed Speed. Refer to the Programming Chapter.

### 6.3.4 Maximum Speed Indicator



This displays the current maximum speed setting.

# 6.3.5 Digital Speed Display

1.55mph

This displays the actual speed of the wheelchair derived from the motors. The display can be set to mph or km/h.


These adjustments can be made in the OBP Menu.

### 6.3.6 Latched



When the control system is operating in a latched condition this symbol will be displayed.

# 6.3.7 Inhibit

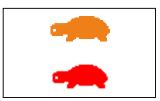

If the speed of the wheelchair is being limited; for example, by a raised seat, then this orange symbol will be displayed.

If the wheelchair is being inhibited from driving, then this red symbol will be flashing.

#### :: Mode Screens ::

### 6.3.8 Actuator Mod€



Displays the sections of the chair currently selected for movement, the name given to the selection and a direction arrow showing what sort of movement is available.

### 6.3.9 Bluetooth Mode



When Bluetooth Mode is entered the following screen will be displayed.

# 6.4 Message Window




The R-net displays warning icons and informational messages, in a dedicated message window.

### 6.4.I Restart



When the control system requires a reboot; for example, after a module re-configuration, this symbol will be flashed.

### 6.4.2 Timer

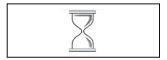

This symbol is displayed when the control system is changing between different states. An example would be entering into Programming Mode. The symbol is animated to show the sands falling.


### 6.4.3 SIEED



This symbol will be displayed for a short time before the R-net enters into a sleep state.

#### 6.4.4 Cross & Tick


These symbols will be displayed during configuration procedures.

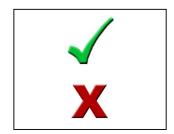

Process completed correctly.

Process not completed correctly.

6.4.5 E-stop

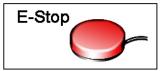

If the External Profile Switch is activated during drive, or actuator operation, this symbol will be displayed.

# 6.4.6 Joystick Displaced



If you operate the Joystick before or just after you switch the control system on, the screen will flash the joystick displaced screen.

You must release and center the Joystick to resume normal operation. If you do not release the Joystick within five seconds the wheelchair will not be able to move, even if you release the Joystick and operate it again. The screen will display a diagnostic screen at this time. You can reset this condition by switching the control system off and on again.


### 6.4.7 Control System Locked



The Control System can be locked in one of two ways. Either using a sequence of deflections and presses with a Joystick or with a physical Key. How the Control System is locked depends on how the wheelchair manufacturer has programmed it.

Refer to Section 5 for a detailed description of the Locking and Unlocking procedures.

### 6.4.8 Diagnostic Screen

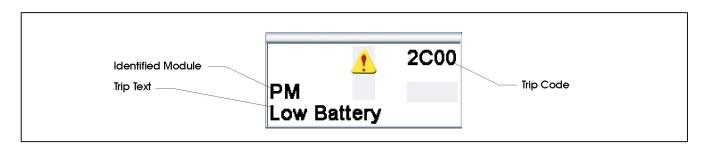


When the control system safety circuits have operated and the control system has been prevented from moving the wheelchair a diagnostics screen will be displayed.

This indicates a system trip, i.e. the R-net has detected a problem somewhere in the wheelchair's electrical system.

Refer to section 5 for a detailed description of Diagnostic screen and procedure. Refer to Chapter 5 Diagnostics for a complete description of the Trip Texts.

# 7 Getting Ready to Drive


• Operate the On/Off switch. The screen will go through an initializing process then show the base screen as follows. In the case on an LED Joystick Module the battery gauge will illuminate.

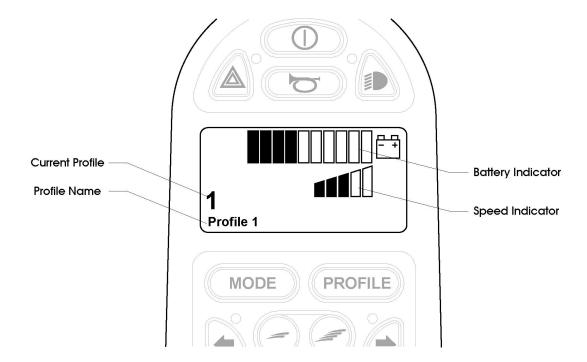


- Check that the Speed Setting is at a level that suits you.
- Push the joystick to control the speed and direction of the wheelchair.



If you push the joystick before or just after you switch the control system on, the screen will flash the joystick displaced screen. You must release and center the joystick to resume normal operation. If you do not release the joystick within five seconds the wheelchair will not be able to move, even if you release the joystick and push it again. The screen will display the diagnostic screen at this time. You can reset this condition by switching the control system off and on again.

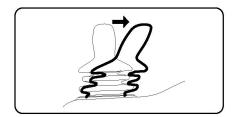


If you do not push the joystick as you switch the wheelchair on and the diagnostic screen is displayed, as in the following diagram, then the R-net has detected a problem somewhere in the wheelchair's electrical system.

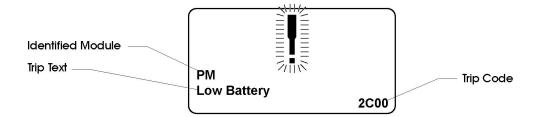

# 8 Settings Menu

The Settings Menu allows the user to adjust the CJSM display in terms of clock adjustment and display format, the brightness of the backlight, the background color and the behavior of the odometer. The menu is accessed by depressing the Speed Down and Speed Up buttons simultaneously. A typical Settings Menu display would be as below.

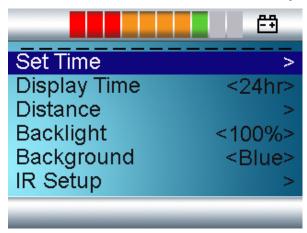

Each of the menu items are described in the following sections.

## 8.I Set Time

A right joystick deflection will enter a clock adjustment screen in which further joystick deflections are used to set the time.

# 8.2 Display Time

This sets the format of the time display or turns it off.

The options are 12hr, 24hr or Off. Left and right joystick deflections are used to change between the options.

#### 8.3 Distance

This sets the functionality of the odometer and a screen as below will appear.



Total Distance This is a value held in the Power Module and relates to the total distance driven using that Power

Module.

Trip Distance

This is a value held in the CJSM and relates to the total distance driven since the last reset.

Display Distance

Sets whether Total Distance or Trip Distance appears as the odometer display on the CJSM.

Clear Trip Distance A right joystick deflection will clear the Trip Distance value.

Exit A right joystick deflection will return to the Settings Menu.

# 8.4 Backlight

This sets the intensity of the LCD backlight.

The adjustable range is 0% to 100% in steps of 10%. Adjustments are made with left and right joystick deflections.

#### 8.5 Background

This sets the color of the screen background. Blue is the standard, but in very bright sunlight then a white background will make the display more visible.

The options are Blue, White and Auto. Left and right joystick deflections are used to change between the options.

Blue means the background will be blue in all Profiles.

White means the background will be white in all Profiles.

Auto means the color will be set by the programmable parameter, **Background**, which can be set to be different across the Profiles. For example, blue for the slower Profiles that are for indoor use and white for the faster Profiles intended for outdoor use. For more details of the parameter, **Background**, refer to the relevant section in the Programming chapter.

### 8.6 IR Setup

IR Set up allows the user access the Omni IR (Infra Red) menus. For full details on how to learn, delete and use the IR functions available please refer to the R-net Omni Technical Manual SK78813.

### 8.7 Exit

Exits the Settings Menu back to normal operation.

SK7798I/6 3I

# 9 Tips for Using Your Control System

# 9.I Driving - General

Make sure that the control system is mounted securely and that the joystick position is correct. The hand or limb you use to operate the joystick should be supported, for example by the wheelchair arm pad. Do not use the joystick as the sole support for your hand or limb - wheelchair movements and bumps could upset your control.

#### 9.2 Driving Technique

The control system interprets your joystick movements and produces appropriate movements of your wheelchair. You will need very little concentration to control the wheelchair, which is especially useful if you are inexperienced. One popular technique is to simply point the joystick in the direction you want to go. The wheelchair will "home-in" on the direction you push the joystick.

The further you push the joystick away from the rest position, the faster the wheelchair will go. Releasing the joystick will stop the wheelchair.

The intelligent speed control system minimizes the effects of slopes and different types of terrain.



The wheelchair user must be capable of driving a wheelchair safely. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

### 9.3 Slow or sluggish movement

If the wheelchair does not travel at full speed or does not respond quickly enough, and the battery condition is good, check the maximum speed setting. If adjusting the speed setting does not remedy the problem then there may be a non-hazardous fault. Contact your service agent.

# IO Precautions for Use



In the event of the wheelchair moving in an unexpected way RELEASE THE JOYSTICK. This action will stop the wheelchair under any circumstances.

### IO.I Hazards

Do not drive the wheelchair:

- Beyond restrictions indicated in your wheelchair user manual, for example maximum inclines, curb height etc.
- In places or on surfaces where a loss of wheel grip could be hazardous, for example on wet grassy slopes.
- If you know that the control system or other crucial components require repair.



Although the R-net control system is designed to be extremely reliable and each unit is rigorously tested during manufacture, the possibility of a system malfunction always exists (however small the probability). Under some conditions of system malfunction the control system must (for safety reasons) stop the chair instantaneously. If there is any possibility of the user falling out of the chair as a result of a sudden braking action, it is imperative that a restraining device such as a seat belt is supplied with the wheelchair and that it is in use at all times when the wheelchair is in motion. PGDT accepts no liability for losses of any kind arising from the unexpected stopping of the wheelchair, or arising from the improper use of the wheelchair or control system.



Do not operate the control system if the chair behaves erratically, or shows abnormal signs of heating, sparks or smoke. Turn the control system off at once and consult your service agent. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Electronic equipment can be affected by Electro Magnetic Interference (EMI). Such interference may be generated by radio stations, TV stations, other radio transmitters and cellular phones. If the chair exhibits erratic behavior due to EMI, turn the control system off immediately and consult your service agent. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



It is the responsibility of the chair manufacturer to ensure that the wheelchair complies with appropriate National and International EMC legislation. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The wheelchair user must comply with all wheelchair safety warnings. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

# II Safety Checks

The electronic circuits in your control system have been designed to be extremely safe and reliable. The on-board microcomputer carries out safety checks at up to 100 times per second. To supplement this safety monitoring you should carry out the following periodic checks.

If the control system fails any of these checks, do not use the wheelchair and contact your service agent.

### II.I Daily Checks

Joystick: With the control system switched off, check that the joystick is not bent or damaged and that it

returns to the center when you push and release it. If there is a problem do not continue with the

safety checks and contact your service agent.

#### II.2 Weekly Checks

Parking brake: This test should be carried out on a level floor with at least one meter clear space around the

wheelchair.

Switch on the control system.

Check that the screen remains on after initialization and that the battery gauge is displaying a

reasonable amount of charge.

Push the joystick slowly forwards until you hear the parking brakes operate. The chair may start to

nove.

Immediately release the joystick. You must be able to hear each parking brake operate within a few

seconds.

Repeat the test a further three times, pushing the joystick slowly backwards, left and right.

Connectors: Make sure that all connectors are securely mated.

Cables: Check the condition of all cables and connectors for damage.

Joystick gaiter: Check the thin rubber gaiter or boot, around the base of the joystick shaft, for damage or splitting,

check visually only, do not handle the gaiter.

Mounting: Make sure that all the components of the control system are securely mounted. Do not overtighten

any securing screws.

# II.3 Servicing

To ensure continued satisfactory service, we suggest you have your wheelchair and control system inspected by your service agent after a period of 1 year from commencement of service. Contact your service agent for details when the inspection is due.

# 12 Battery Gauge

The battery gauge is included to let you know how much charge is left in your batteries. The best way for you to use the gauge is to learn how it behaves as you drive the wheelchair. Like the fuel gauge in a car, it is not completely accurate, but it will help you avoid running out of "fuel".

The battery gauge works in the following way:

When you switch on the control system, the battery gauge shows an estimate of the remaining battery charge.

The battery gauge gives you a more accurate reading about a minute after you start driving the wheelchair.



When you replace worn out batteries, fit the type recommended by the wheelchair manufacturer. If you use another type the battery gauge may be inaccurate.

The amount of charge in your batteries depends on a number of factors, including the way you use your wheelchair, the temperature of the batteries, their age and the way they are made. These factors will affect the distance you can travel in your wheelchair. All wheelchair batteries will gradually lose their capacity as they age.

The most important factor that reduces the life of your batteries is the amount of charge you take from the batteries before you recharge them. Battery life is also reduced by the number of times you charge and discharge the batteries.

To make your batteries last longer, do not allow them to become completely flat. Always recharge your batteries promptly after they are discharged.

If your battery gauge reading seems to fall more quickly than usual, your batteries may be worn out.

### 12.1 How to Read a Battery Gauge

If the battery gauge shows red, yellow and green, the batteries are charged. (Bars 1-10)

If the battery gauge shows just red and yellow, then you should charge the batteries as soon as you can. (Bars 1 - 7)

If the battery gauge shows just red, either steady or flashing slowly, then you should charge the batteries immediately. (Bars 1 – 3)




Do not operate the control system if the battery is nearly discharged. Failure to comply with this condition may leave the user stranded in an unsafe position, such as in the middle of a road. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

# **I3** Battery Charging

To charge the wheelchair batteries connect the charger plug into the battery charger socket on the R-net JSM. You will not be able to drive the wheelchair when the charger is connected.

To connect the charger plug, ensure the single pin is at the bottom, as shown in the following illustration, then offer the charger plug to the R-net in a horizontal orientation. The molded guide on the R-net will help you to locate the plug. Ensure the plug is pushed fully in position.

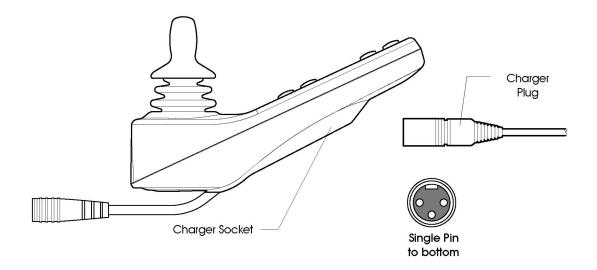



Do not exceed the maximum charging current of I2Arms. Always use an off-board charger fitted with a Neutrik NC3MX plug. Failure to observe these conditions could result in poor contact resistance in the charger connector resulting in overheating of the charger plugs. This presents a potential burn hazard for the user. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Ensure that the charger plug pins are of the correct polarity to be compatible with the pin polarity shown on the control system's specific data sheet. Failure to observe this condition could result in a burn hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Do not disconnect batteries or open-circuit the circuit breaker while charging is in progress. Failure to observe this condition could result in a burns hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Only use the battery charger that has been supplied with your wheelchair. The use of incorrect chargers could damage the batteries, wheelchair, control system or charger itself, or may result in parts overheating creating the potential for burns or even fire. PGDT accepts no liability for losses of any kind if the charger is incompatible with the control system (see Chapter 2, section 7) or any other part of the wheelchair system.

# 14 Programming

The control system can be programmed to meet your needs. Programming can be performed using the OBP (On-board Programming) feature or the specialist R-net software and Dongle or the Diagnostic Test Tool (DTT).

If you re-program your control system, make sure that you observe any restrictions given in your wheelchair user manual. Note any changes you make for future reference.




Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT electronic control systems. Incorrect programming could result in an unsafe set-up of a wheelchair for a user. PGDT accepts no liability for losses of any kind if the programming of the control system is altered from factory preset values.

# 15 Joystick Knobs

The knob fitted to your joystick is suitable for most applications. If you would prefer another type, there is a range of alternatives available. Please contact your wheelchair distributor or manufacturer for advice. Do not replace the joystick knob with any unauthorized item - it may cause hazardous operation.



Do not replace the joystick knob with any unauthorized item It may cause hazardous operation. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

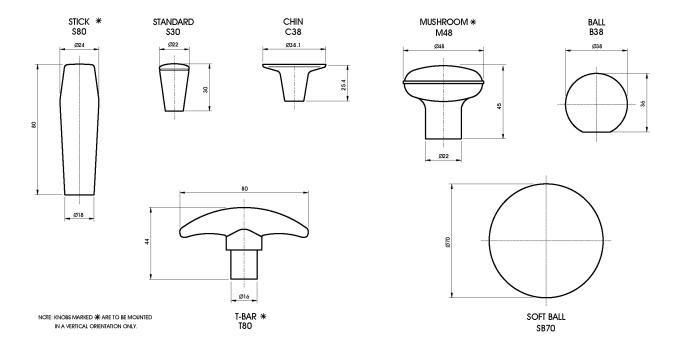

# 16 Servicing

All repairs and servicing must be carried out by authorized service personnel. Opening or making any unauthorized adjustments or modifications to the control system or its components will invalidate any warranty and may result in hazards to yourself or other people, and is strictly forbidden.



PGDT accepts no liability for losses of any kind arising from unauthorized opening, adjustment or modifications to the R-net control system.



If the control system is damaged in any way, or if internal damage may have occurred through impact or dropping, have the product checked by qualified personnel before operating. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

# 17 Warranty

The R-net control system is covered by a warranty period defined by the wheelchair manufacturer. For details of the warranty period, please contact your service agent.

The warranty will be void if the R-net control system has:

- Not been used in accordance with the R-net control system Technical Manual, SK77981.
- Been subject to misuse or abuse.
- Been modified or repaired by non-authorized persons.



The warranty will be void if the R-net has not been used in accordance with Technical Manual SK7798I, the R-net has been subject to misuse or abuse, or if the R-net has been modified or repaired by unauthorized persons.



# **CHAPTER 2 - INSTALLATION**

#### I Documentation

### I.I R-net Operation

Study Chapter 1. It is important that the operation information in Chapter 1 is supplied with the wheelchair, either as part of the wheelchair user handbook or as a separate document.

This Chapter sets out the installation conditions that must be complied with in order to meet the safety requirements of TÜV (Germany), ISO7176-14 and EN12184.

# I.2 Program Settings

The R-net control system is supplied with preset settings that are chosen with the wheelchair manufacturer to ensure safe operation and compliance with relevant legal requirements, over the whole of the operating range of the wheelchair.



It is the manufacturer's responsibility to program the control system to suit the vehicle model and ensure safe operation in compliance with relevant legal requirements over the whole of the operating range. PGDT accepts no liability for losses of any kind due to failure to, or incorrect programming of the R-net Control System. Refer to Chapter 3 for programming details.

The wheelchair must stop within the maximum distance specified for the country in which the wheelchair will be used. TÜV Product Service (Germany) specify the distance to be as stated in EN12184.

If users with particular disabilities need very low braking rates and this results in a longer stopping distance, the maximum speed must be re-programmed so that the stopping distance requirement is satisfied.

State in the wheelchair user handbook that it is the responsibility of the person programming the control system to make sure that the stopping distance requirement is satisfied. If the braking rate is low, the forward and reverse maximum speed settings may need to be re-programmed. To assist the person in this task, include a graph in the wheelchair user handbook showing the relationship between the maximum forward/reverse speed settings and the forward/reverse braking rate which is required to ensure the correct stopping distance.

It may be possible to program settings which compromise the stability of the wheelchair. Perform suitable tests to establish which programming restrictions are needed to prevent instability. State any programming restrictions in the wheelchair user handbook.

State in the wheelchair user handbook that it is the responsibility of the person programming the control system to make sure that the settings are safe and to note any programming changes that they make.



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT electronic control systems. Incorrect programming could result in an unsafe setup of a wheelchair for the user. PG Drives Technology accepts no liability for losses of any kind if the programming of the control system is altered from factory pre-set values. PGDT accepts no liability for losses of any kind if the drive or stability characteristics of the chair are altered without prior notification and discussion with PGDT.



It is possible for wheelchair manufacturers to limit the values of speeds and accelerations that can be programmed in the field. This facility ensures a safe-operating envelope for the wheelchair can be maintained.

# I.3 Soft-Stop

The R-net has a programmable value called Soft Stop Rate that sets the emergency stopping distance. You must ensure that the emergency stopping distance is within the distance specified for the country in which the wheelchair will be used. TÜV Product Services (Germany) specify the distance to be as stated in EN12184.

#### I.4 Other Information

You must provide a diagram in the wheelchair user handbook showing the user controls and the main features of the control system.

In addition, you should include a brief specification of operating supply voltage range and operating temperature range.

# 2 Immobilizing the Wheelchair

#### 2.I Prevention of Unauthorized Use

TÜV requires that the wheelchair must have a means of preventing unauthorized use. This can be implemented as detailed in Chapter 1 section 5.4.

### 2.2 Charger Interlock

ISO 7176-14 requires you to provide a means of preventing the use of the wheelchair while the batteries are being charged. The charger socket and on-board charger connection fitted to PGDT R-net control systems include an inhibit facility. Refer to section 6.5.

Contact PG Drives Technology if you need advice.



The chair manufacturer is responsible for providing a means of preventing the use of the wheelchair while the batteries are being charged. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

# 2.3 Power Down

The control system can be programmed to power down if the joystick is not operated over a set period of time. Refer to Chapter 3 for programming details.

To restart, switch the control system on.


# 3 Connections

The following is a selection of the most common configurations.

# 3.I Control Configurations


# 3.I.I Basic Configuration

Consists of a Power Module, a Communication Cable and a Joystick Module.

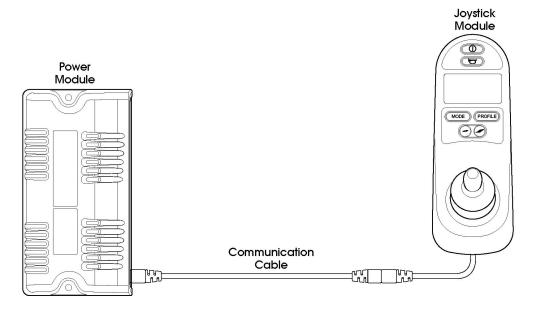


# 3.1.2 Joystick & ISM Configuration

Consists of a Power Module, an Intelligent Seating/Lighting Module (ISM), 2 Communication Cables and a Joystick Module.

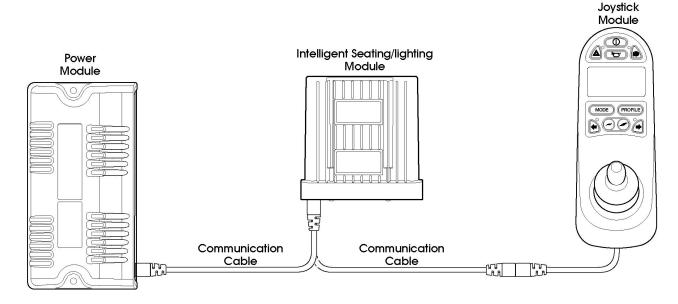

# 4 Mounting

# 4.I Joystick Module Mounting

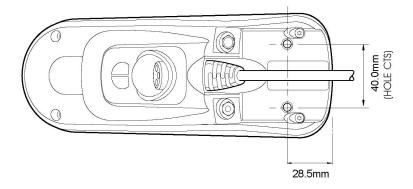


# 4.I.I General

The Joystick Module should be secured using two M5 screws with a maximum penetration of 12mm. Be careful not to overtighten the screws. See data sheet for further information.

### 4.1.2 Orientation

The Joystick Module must be mounted with the joystick shaft pointing vertically upwards. If you want to use any other mounting alternatives then contact PGDT.

# 4.2 Power Module Mounting

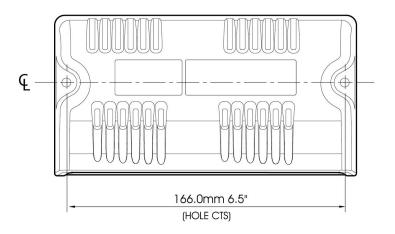

### 4.2.I General

Fix the Power Module to the wheelchair chassis using suitable M5 or equivalent hardware.


#### 4.2.2 Orientation


The function of the Power Module is not sensitive to mounting orientation; however, it should be mounted in such a way that water cannot enter and remain in the connector recesses. It is recommended that the unit is not mounted with the connectors uppermost. The Power Module has an IPX4 moisture resistance rating.

#### 4.2.3 Position

The Power Module must be mounted in a position where it is not exposed to conditions of water or dust above those specified in ISO7176/9. The Power Module is designed to withstand levels of shock and vibration experienced when mounted to the chassis of a wheelchair. Direct impacts onto the unit should be avoided.

### 4.3 ISM Mounting

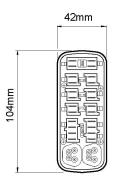

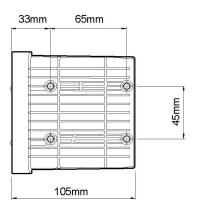

# 4.3.I General

The ISM should be secured using four M5 screws with a maximum penetration of 7mm. Be careful not to over tighten the screws. See data sheet for further information.

#### 4.3.2 Orientation

The function of the ISM is not sensitive to mounting orientation; however, it should be mounted in such a way that water cannot enter and remain in the connector recesses. It is recommended that the unit is not mounted with the connectors uppermost. The ISM has an IPX4 moisture resistance rating.

#### 4.3.3 Position

The ISM must be mounted in a position where it is not exposed to conditions of water or dust above those specified in ISO7176/9. The ISM is designed to withstand levels of shock and vibration experienced when mounted to the chassis of a wheelchair. Direct impacts onto the unit should be avoided.

# 4.4 Cables

The cables to the ISM must be routed and secured in such a way as to prevent damage to them, for example by cutting or crushina.

Contact PGDT if you need further advice.


# 5 Joystick Module Wiring

The Joystick Module is connected to the Power Module with a Communication Cable.

To connect the Communication Cables:

 Holding the connector housing, firmly push the connector into its mate until you can no longer see the yellow plastic.

The connectors are secured using a friction system.

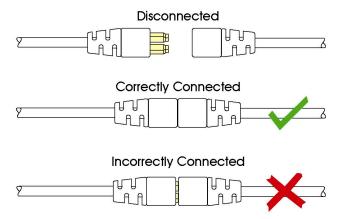

To disconnect the Communication Cables:

Holding the connector housing firmly, pull the connectors apart.



Do not hold or pull on the cable. Always grip the connector when connecting and disconnecting.



When the control system is first switched on after a connection, or system component change the Timer will be displayed whilst the system checks itself and then the Re-start icon will be displayed. Switch the control system off and on again to operate.

Route and secure the cable in such a way as to prevent damage, for example by crushing or cutting.

# 5.I Charger Connector

An Off-Board Charger can be connected to the Joystick Module's charger connector. The charger connector is Neutrik 3 pin type NC3FPP or equivalent, and the maximum charging current is 12A rms. Only chargers fitted with Neutrik NC3MX plugs should be connected into the Joystick Module. The pin connections of the charging socket are as below.

Pin	Connection						
1	Battery Positive						
2	Battery Negative						
3	Inhibit						





The maximum permissible charging current is I2Arms.

# 6 Power Module & ISM Wiring

#### 6.I General

Study the data sheets for the Power Module and the ISM to identify:

- The output current, ratings and restrictions.
- The connector pin assignments.

Recommendations for the cross-sectional area, ratings and materials for wiring are given in the table in section 6.2. These depend on the application. You are responsible for establishing the suitability of the particular wiring arrangement used on the wheelchair. PGDT can make general recommendations for wiring to R-net control systems, but PGDT accepts no responsibility for the wiring arrangement used.

Make sure that the connectors you use are reliable under all operating conditions and correctly wired with no short circuits. Do not use unsuitable components - it may result in poor wheelchair reliability.



The chair manufacturer is responsible for establishing the suitability of the particular wiring arrangements used on the wheelchair, for both normal use and stalled conditions. PGDT can make general recommendations for wiring for R-net control systems, but PGDT accepts no responsibility for, and accepts no liability for losses of any kind arising from, the actual wiring arrangement used.



The chair manufacturer is responsible for ensuring that only the mating connectors specified by PGDT on the control system's specific data sheet are used to connect to the control system. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The chair manufacturer is responsible for ensuring that suitable connectors are used and securely mated throughout the chair wiring system and that the workmanship associated with the wiring system is of a good enough quality. Failure to meet this condition could result in intermittent operation, sudden stopping or veering, or even create a burn or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

The power connectors for the R-net Power Module are specially designed by PGDT for wheelchair applications. The crimps are standard AMP Timer parts but the plastic housings are custom.

These connectors can be purchased from PG Drives Technology or from Inconnect directly. Inconnect will also provide a preassembled cable service. Contact details for Inconnect are as follows.

Inconnect Tel: +44 (0) 1522 783030

Fax: +44 (0) 1522 783031


www.inconnect.uk.net

The connector part numbers are as shown in the following table:

Connector	PG Drives Technology Reference:	Inconnect Reference:
Battery (Up to 6mm²)	D49712 PG80-B	IPG-5202
Motor/Brake (Up to 4mm²)	D49713 PG80-M	IPG-5401
Battery (Up to 10mm²)	D50287 PG80-B-HP	IPG-5205-PS
Motor/Brake (Up to 10mm²)	D50947 P G80-M-HP	IPG-5405-PS
OBC	D50752 PG-OBC	IPG-7301-PS
Inhibit	D50753 PG-Inhibit	IPG-8202-PS
Actuator	D50754 PG-Actuator	IPG-8201-PS
Lighting (both sides)	D50784 PG-Lights	IPG-9401-PS

# 6.I.I Power Module Wiring


The following diagram gives details of the Power Module connections.

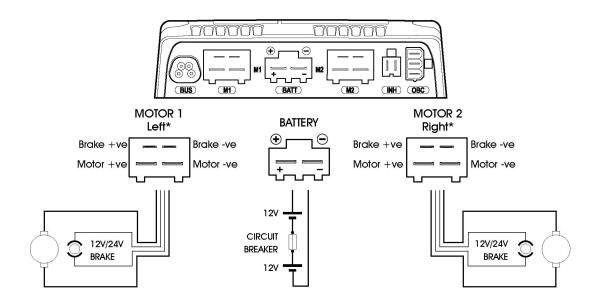

\* Assumes no Motor Swap programming has been undertaken

Joystick Movement		M1		12	Actual Movement	
Forward	Motor +ve		Motor +ve		Forward	**
roiwaia	+ve	-ve	+ve	-ve	roiwaia	^ ^
Backward	-ve	+ve	-ve	+ve	Backward	**

\*\* Assumes no Joystick Orientation, Invert M1 Direction or Invert M2 Direction programming has been undertaken

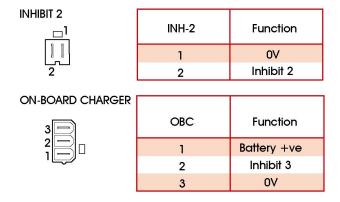


The Power Module is shipped with rubber bungs inserted into some of the connectors.

Only remove the rubber bung from the required connectors.

# 6.1.2 ISM Wiring

The following diagram gives details of the ISM connections.

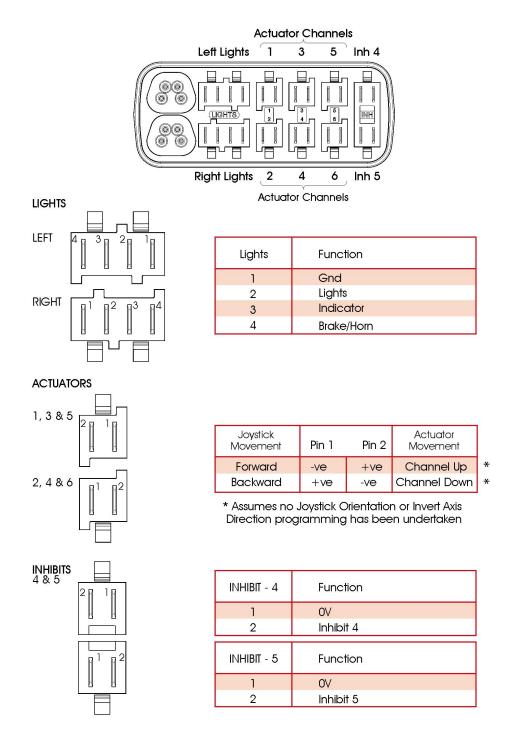

The ISM is shipped with rubber bungs inserted into some of the connectors.

Only remove the rubber bung from the required connectors.

#### 6.1.3 Crimping

Good quality crimping is essential in ensuring the long term reliability of the wheelchair's electrical system. Poor quality crimps may initially appear to be satisfactory but, over time, they may cause problems. It is recommended that crimp quality is maintained by implementing the procedures detailed in IEC-60352-2 1990.



Defective or poor quality crimps may affect the warranty of the control system. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

Hand tools for crimping and extraction are available from Inconnect. The references are as below.

### Battery, Motor and Brake wiring

Small crimp tool handle (up to 6.0mm²): ICT-29020

Large crimp tool handle (up to 6.0mm²): ICT-29030

Die set for 0.5-1.5mm² wire: ICD-29468

Die set for 2.5-4.0mm² wire: ICD-29465

Die set for 4.0-6.0mm² wire: ICD-29467

Crimp tool including die set (8-10.0mm²): AMP Part No. 734533-1

Extraction tool for 0.5-1.5mm<sup>2</sup>: IET-503
Extraction tool for 2.5-10.0mm<sup>2</sup>: IET-552

OBC, Inhibit, Actuator and Lighting wiring

Small crimp tool handle (up to 6.0mm²): ICT-29020

Die set for 0.5-1.5mm² wire: ICD-29468

Extraction tool for 0.5-1.5mm²: IET-503



Only use the exact tools as specified.

For details of automatic crimp tools contact Inconnect.

#### 6.2 Wire Gauge and Types

The table below gives the minimum recommended wire sizes for various R-net Power Module and ISM specifications.

Power Module	Battery	Motor	Brakes	Actuators *	Lighting	Inhibit
60A	6.0mm <sup>2</sup>	4.0mm <sup>2</sup>	0.5mm <sup>2</sup>	1.5mm <sup>2</sup>	0.5mm <sup>2</sup>	0.22mm <sup>2</sup>
80/90A	6.0mm <sup>2</sup>	4.0mm <sup>2</sup>	0.5mm <sup>2</sup>	1.5mm <sup>2</sup>	0.5mm <sup>2</sup>	0.22mm <sup>2</sup>
120A	8.0mm <sup>2</sup>	8.0mm <sup>2</sup>	0.5mm <sup>2</sup>	1.5mm <sup>2</sup>	0.5mm <sup>2</sup>	0.22mm <sup>2</sup>

These recommendations are for cable lengths of less than 1000mm. They are derived from well proven field experience of various international wheelchair manufacturers.

<sup>\*</sup>If lower current limits are programmed for a particular ISM channel(s), it is possible to use smaller gauge wire. The following recommendations are made.

Maximum Up/Down Current Limit	Minimum Wire Size						
3A	0.5mm <sup>2</sup>						
5A	0.8mm <sup>2</sup>						
10A	1.0mm <sup>2</sup>						



Battery and motor wires should have IO5°C rated PVC insulation.



The chair manufacturer is responsible for establishing the suitability of the particular wiring arrangements used on the wheelchair. PGDT can make general recommendations for wiring for R-net control systems, but PGDT accepts no responsibility for, and accepts no liability for losses of any kind arising from, the actual wiring arrangement used.

### 6.3 Battery Connections

The control system incorporates sophisticated current limiting circuitry as protection for the circuits in the control system.

ISO 7176-14 requires you to provide protection against short circuits in the battery wiring and the power loom or the extremely unlikely event of a short circuit in the control system.

Place a suitable circuit breaker in series with the battery supply, for example in the link between two 12V batteries. If your batteries are held in separate enclosures, you must provide a circuit breaker with each of them.

The rating of the circuit breaker must match the capacity of the wiring specified in section 6.2. The following table gives some recommendations for circuit breaker selection. These recommendations are derived from well proven field experience of various international wheelchair manufacturers. Nevertheless, manufacturers must confirm these recommendations by carrying out suitable tests.

Power Module	Circuit Breaker Value
60A	50A
80/90A	90A
120A	90A

ISO 7176-14 states that the minimum operating time for the circuit breaker when the wheelchair is stalled is 15 seconds.



The chair manufacturer must install a suitable circuit breaker to provide protection against short circuits in the battery wiring, power loom or the control system. Failure to comply with this could result in a fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

### 6.4 Motor Connections

If a circuit breaker is fitted in series with a motor, it is essential that the wheelchair assume a safe condition the moment the circuit breaker operates. You must therefore fit a circuit breaker with an auxiliary switch that disconnects the solenoid brake from the Power Module.

#### 6.5 Inhibit Connections

The Power Module contains the Inhibit 2 and 3 connections, and the ISM contains the Inhibit 4 and 5 connections. These connections can be used to limit speed, inhibit drive and /or inhibit actuator channels. Refer to Chapter 3 – Programming for full details.

SK7798I/5 5I

#### 6.6 Lighting Connections

Each lighting output is rated at 21W maximum. Use suitably rated wire to each bulb. The output is self-protecting against wiring and bulb faults and will shut-off if the maximum power rating is exceeded. Likewise, if the output becomes too hot it will shut-off to prevent permanent damage to itself.

#### 6.7 Indicator Connections

Each indicator output is rated at 21W maximum. Use suitably rated wire to each bulb. The outputs are self-protecting against wiring and bulb faults and will shut-off if the maximum power rating is exceeded. Likewise, if either output becomes too hot it will shut-off to prevent permanent damage to itself.

If an indicator bulb fails, the ISM will detect this and flash the remaining bulb for that side at 3Hz as well as sending information to the Joystick Module so that the relevant indicator LED flashes likewise.

#### 6.8 Actuator Connections

Each actuator channel is rated at 15A maximum, although only 12A is available when the channel is programmed to 100% speed. If currents of greater than 12A are required, the programmed speeds for that channel must be reduced. The level of reduction is dependent upon the electrical characteristics of the actuator motor, so an empirical process to establish the level of the reduction is required.

If the wheelchair is fitted with solid mechanical end-stops at the end of the actuator travel, it is not necessary to fit limit switches as the ISM can detect when the actuator motor has stopped and will cut off the power to it.

#### 6.9 On-Board Charger Connections

The Battery +ve and 0V connections are rated at 12A rms, Use suitably rated wire for these connections.



The maximum permissible charging current is I2A rms.

### 7 Batteries

The control system is designed for operation with 24V lead acid batteries. The batteries may be wet or gel electrolyte types. Contact the manuafacturers for advice on battery selection.

#### 7.I Battery Charging

The R-net system can support two methods of charging, Off- Board and On-Board.

An On-Board Charger should be connected to the Power Module via the OBC connector, refer to the wiring diagram of the Power Module for pin details and to the section On-Board Charger Connections for details of maximum current.

An Off-Board Charger can be connected through a Charger Socket found on either a Joystick Module or the Omni. The charging socket is Neutrik 3 pin type NC3FPP or equivalent, and the maximum charging current is 12A rms. Only chargers fitted with Neutrik NC3MX plugs should be connected into the Joystick Module.



The maximum permissible charging current for both methods is I2A rms.

To prevent the wheelchair from driving whilst the charger is connected, pin 3 must be linked to pin 2 inside the charger's plug.



Do not exceed the maximum charging current of I2A rms. Always use an Off-Board Charger fitted with a Neutrik NC3MX plug. Failure to observe these conditions could result in poor contact resistance in the charger connector resulting in overheating of the charger plugs. This presents a potential burn hazard for the user. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Ensure that the charger plug pins are of the correct polarity to be compatible with the pin polarity shown on the control system's specific data sheet. Failure to observe this condition could result in a burn hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Do not disconnect batteries or open-circuit the circuit breaker while charging is in progress. Failure to observe this condition could result in a burns hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

# 8 Drive Motors

The control system is designed to be connected to permanent magnet DC motors, fitted with suitable gearboxes and solenoid brakes.

In order to optimize the performance of the wheelchair, the control system must be matched to the motor terminal impedance. This matching is implemented by programming the control system. The parameter for adjustment is Compensation. Refer to Chapter 3.

The Compensation value should be set in accordance with the armature resistance of the motor and all cables and connectors between the R-net and the motor.



The chair manufacturer is responsible for ensuring that the control system is matched to the motor resistance. Failure to do this may result in poor control characteristics, which in extreme instances can make a chair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The chair manufacturer is responsible for always ensuring that any replacement motors or gearboxes are fully compatible with the originals that the control system was designed to match. Failure to do this may result in poor control characteristics, which in extreme instances can make a chair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Users must not move a control system from one chair type to install it on a different chair type. Control systems with different part numbers may have both hardware and software differences to ensure that they are compatible with the electrical and dynamic characteristics of their specific target vehicles. The characteristics of one type of control system may not be compatible with a different, unauthorized chair. Failure to observe this warning could result in an unsafe setup for the wheelchair user and may create a fire hazard depending on the motors, wiring, connectors and circuit breakers installed on the unauthorized chair. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

# 9 Solenoid Brakes

The solenoid brakes can be either 12V or 24V. However, the control system must be programmed for the correct voltage. Refer to data sheet for the programming details of the control system. Refer to chapter 3 for information on programming.

If the current to either brake is less than  $200\mu A$  or greater than 1A, a brake fault will be detected.



It is essential that the control system is programmed for the correct brake voltage. If it is not, damage may occur to the brakes or the drive performance of the wheelchair may be affected.

# IO Programmer Connection

### IO.I OBP Programming

On-Board Programming (OBP) allows the wheelchair to be programmed, via the Joystick Module, to suit an individual user.


OBP can be entered via two methods – connection of a hardware dongle or a keycode. The actual method required is set by the factory programming of the control system.




OBP is not possible using an LED Joystick Module.

If the hardware dongle method is set, then a PGDT OBP Dongle needs to be connected to allow access to OBP. To use the dongle, follow the sequence below:

- Turn off the control system.
- Insert the R-net Dongle along the communication cables in the system configuration.

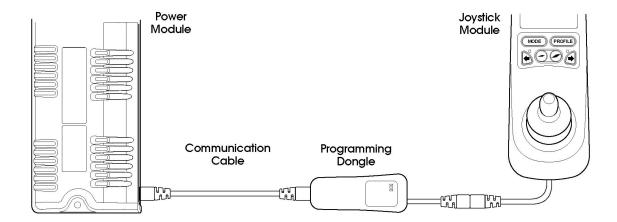

- Turn on the control system
- After initialization press the Mode key until the OBP screen is reached. See following diagram.

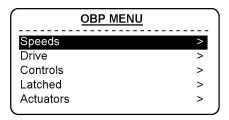

After initialization OBP will be automatically entered.

If the keycode method is set, then the following button sequence will allow entry to OBP. Note, a dongle will also allow OBP access if this method is set.

- Hold down the Horn button and then hold down the On/Off button until there is a short bleep. Note, the normal power-up sound will occur prior to this bleep.
- Release the Horn button, but continue to hold down the On/Off button until there are two further short bleeps.
- Release the On/Off button, there will now be a longer bleep and OBP mode will be entered. See previous diagram.




If the keycode method is set, then it may be possible for a wheelchair user to gain entry to the OBP function. PGDT accepts no liability for losses of any kind if this method of OBP is set. PGDT also accepts no liability for losses of any kind if OBP access is gained by unauthorised personnel obtaining a dongle.

#### IO.I.I Navigating the OBP Screens

Whilst in OBP deflecting the joystick to the left, whilst the parameter title is highlighted, will display an elongated version of the parameter name.

There are two basic forms of display screen – one which shows global parameters and one which shows parameters which are profile dependent, i.e. can have different values in each of the drive profiles.

Example of a global parameter screen:

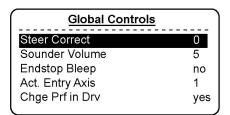

Forward joystick deflections will select the parameter above.

Reverse joystick deflections will select the parameter below.

Left joystick deflections will take you to the previous screen.

Right joystick deflections will have no effect.

Example of a profiled parameter screen:

(	FWD	Pr1	Pr2	Pr3	Pr4	
	Acc ▲	30	35	40	45	
	Acc ▼	20	20	20	20	
	Dec▲	40	40	45	50	
	Dec▼	30	30	30	30	
	Spd▲	70	80	90	100	
l	Spd▼	20	25	30	20	

Forward joystick deflections will select the parameter above.

Reverse joystick deflections will select the parameter below.

The first left joystick deflection will display the parameter name in full.

The second left joystick deflection will take you to the previous screen.

Right joystick deflections will select the parameter to the right, if available.


#### IO.I.2 Making an Adjustment

Once the parameter for adjustment has been selected, the Speed buttons can be used to make adjustments.

To ensure all new values are stored correctly, exit OBP mode with a series of left hand joystick deflections

#### IO.2 PC Programming

To utilize the R-net PC Programming Suite the R-net Dongle must first be connected in the communications system as shown in section 10.1. A USB cable can then be connected between the Dongle and a PC with the R-Net PC Programmer installed. For further details on programming refer to Chapter 3.

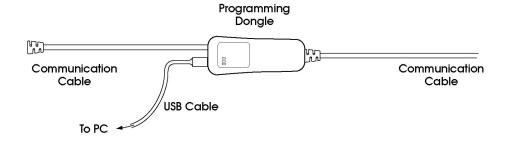

### IO.3 DTT Programming

The PGDT Diagnostic Test Tool may be used to program the Control System. To utilize a DTT:

- Switch off the Control System.
- Insert DTT into the communications system using the R-net Cable supplied.
- Switch on the Control System.

For detailed information about the DTT refer to the Diagnostic Test Tool Technical Manual SK79393.

# II Production Tests

• Perform the following tests, in order, on each wheelchair before dispatch.

These tests should be conducted in an open space and a restraining device such as a seat belt should always be used. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

### II.I Mounting

• Make sure that all the control system is securely mounted. Do not overtighten any securing screws.

# II.2 Cables and Connectors

Check all cables and connectors for damage. Make sure that all connectors are securely mated.

# II.3 Joystick and Gaiter

- Check that the joystick is not bent or damaged.
- Check the thin rubber gaiter or boot, around the base of the joystick shaft, for damage or splitting. Check visually
  only, do not handle the gaiter.

Check that the joystick returns to the center position when you push and release it.

### II.4 Preset Settings

Make sure that the control system is using the correct settings. Refer to the Chapter 3 for detailed instructions.

Control Systems are always supplied with the settings shown on the relevant data sheet.

#### II.5 Operational Test

This test should be carried out on a level floor with at least one meter clear space around the wheelchair.

- Switch on the control system.
- Check that the battery gauge remains on, or flashes slowly, after one second.
- Push the joystick slowly forwards until you hear the parking brakes operate. The chair may start to move.
- Immediately release the joystick. You must be able to hear each parking brake operate within a few seconds.
- Repeat the test a further three times, pushing the joystick slowly backwards, left and right.

#### II.6 Test Drive

Drive the wheelchair and make sure that it operates correctly for all positions of the user controls.

# II.7 Soft-Stop Test

- Drive the wheelchair at full forward speed and switch the control system off.
- The wheelchair must not stop suddenly, but should decelerate to standstill.

In addition, ensure that the requirements in section 1.3 of this chapter are satisfied.

### II.8 Lights, Indicators and Hazard Lamps Test

If lights are fitted:

- Visually check each bulb for correct illumination.
- Check each bulb for correct illumination and that the flashrate is 1.5Hz  $\pm$  0.5Hz.
- Disconnect each bulb in turn and check that the remaining bulb for that side flashes at 3Hz ± 0.5Hz.

If hazard lamps are fitted:

• Check each bulb for correct illumination and that the flashrate is 1.5Hz  $\pm$  0.5Hz.

### II.9 Actuator Test

If actuators are fitted:

- Check each motor for correct direction of movement.
- Ensure the mechanical end-stops are secure and that they stall the actuator motors, thus operating the ISM's automatic end-stop detection.

# II.IO Inhibit Input Test

Insert an appropriate battery charger, or equivalent inhibit connection, into the Charger Connector on the JSM and ensure that drive is inhibited.

If Inhibits 2, 3, 4 and 5 are used for inhibit or speed limiting purposes, suitable tests should be devised to ensure correct operation.

# I2 Electromagnetic Compatibility (E.M.C.)

The R-net family has been tested for compliance with EC directive 89/336/EEC, and the E.M.C. requirements of EN12184, the FDA and the FCC. The guidelines in this section will help you to make sure that your wheelchair installation will easily meet these requirements. You should consider E.M.C. and perform relevant tests as early as possible in the design phase.

#### 12.1 Emissions

A typical wheelchair and R-net installation have been type tested and have passed the requirements of CISPR 22 and FCC CFR47 part 15.

Observe the following recommendations to minimize radio frequency emissions:

### 12.1.1 Motor Suppression

For the drive motors solder a suitable suppression capacitor between the brush holders, inside the motor cases. Keep the capacitor's lead length as short as possible. We recommend a value of 4n7F 250V AC ceramic. The maximum value you should use is 10nF. A typical type is Roderstein WY0472MCMCF0K.

The above should also be applied to actuator motors, if fitted. If it is not possible to fit these capacitors inside the actuator motors, then contact the motor manufacturers for further advice. Alternatively, placing the capacitor externally, but as close as possible to the motor, may have the same effect.

Some actuators have their own EMC suppression components fitted internally. In these instances, it may not be appropriate to introduce further capacitance. If you are uncertain, contact the actuator manufacturers for further advice.

#### 12.1.2 Cables

You do not need to use screened battery and motor looms, but:

- Keep the length of all wiring to a minimum.
- Make sure the loop area of the wiring is minimized. Route the positive and negative wires to each motor together.
   Route the battery positive and negative wires together. Where possible, route the battery and motor looms together. If fitted, route the lighting, indicator and actuator wires together.
- Secure all looms to the wheelchair's chassis over as much of their length as is practical.
- Do not use the control system connectors as junction points for the battery connections. Separate junction points away from the Power Module should be provided for the other wheelchair electrical functions.

### 12.2 Immunity

The R-net control system family has been stringently tested for susceptibility to electromagnetic radiation over the frequency range 26MHz to 1GHz. The installations passed the FDA requirements and the proposed requirements of EN12184.

Follow the recommendations in section 12.1.2 to ensure maximum immunity to electromagnetic radiation.

#### 12.3 Electro-Static Discharge (E.S.D.)

There are various international standards currently under development for this aspect of the system's performance. At present, most of the standards are specifying the system to be tested to requirements of IEC1000-4-2. Tests are carried out at 8kV air discharge (to nonconductive surfaces) and 6kV contact discharge (to conductive surfaces). The R-net family has been tested to these levels.

If you need advice please contact PGDT.

# **I3** Battery Gauge

Refer to Chapter 1 for how to read the battery gauge.

The battery gauge typically starts to flash slowly when the battery voltage falls below 23.3V whilst the wheelchair is driving on a level surface.

For optimum accuracy of the battery gauge and low battery indicator, the control system should be programmed with the approximate nominal capacity of the wheelchair battery. However, accuracy is not greatly affected if the programmed type and capacity do not closely match the battery.

The most important factor affecting the accuracy of the battery gauge is the resistance of the cable and connections between the battery and the control system. The control system must be matched approximately to the cable resistance of your wheelchair to make the battery gauge accurate. The parameter for adjustment is Cable Resistance, refer to Chapter 3.

As a guide, 2.5mm<sup>2</sup> cable has a resistance of about 8 milliohms per meter; 4.0mm<sup>2</sup> cable has about 5 milliohms per meter and 6.0mm<sup>2</sup> has about 3.3 milliohms per meter. Circuit breakers and connectors usually account for about 15 milliohms.

These values will be chosen at the time the control system is being specified by the wheelchair manufacturer. Like the preset acceleration rates, once the values for the battery are decided, they are programmed into control systems during manufacture and should never need changing.

If you need advice, contact PGDT.



# **CHAPTER 3 - PROGRAMMING**

SK7798I/5 6I

# I Introduction

This chapter gives an overview of the programmable parameters within the R-Net control system. The R-Net can be programmed using either the On-board Programming (OBP) facility, the Diagnostic Test Tool (DTT) or a PG Drives Technology R-Net PC Programmer.



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT control systems. Incorrect programming could result in an unsafe set-up of a wheelchair for a user. PGDT accepts no responsibility for losses of any kind if the programming of the control system is altered from the factory pre-set values.

# 2 On-Board Programming

On-Board Programming (OBP) allows the wheelchair to be programmed, via the Joystick Module, to suit an individual user. There is no need for any additional programming hardware. While in OBP mode, the joystick and the speed control buttons are used to navigate the menus and make parameter adjustments.

If a parameter is available in OBP mode the title used on the LCD will be listed, in brackets, next to the Parameter Title in this chapter.

Example: 6.9 Maximum Forward Deceleration - (Fwd Decel Max)

### 2.I How to enter OBP Mode


OBP in the field can be entered via two methods – connection of a hardware dongle or a keycode. The actual method required is set by the factory programming of the control system.



OBP is not possible using an LED Joystick Module.

If the hardware dongle method is set, then a PGDT OBP Dongle needs to be connected to allow access to OBP. To use the dongle, follow the sequence below:

- Turn off the control system.
- Insert the R-net Dongle along the communication cables in the system configuration.
- Turn on the control system.
- After initialization press the Mode key until the OBP screen is reached. See following diagram.

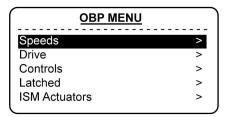

If the keycode method is set, then the following button sequence will allow entry to OBP. Note, a dongle will also allow OBP access if this method is set.

- Hold down the Horn button and then hold down the On/Off button until there is a short bleep. Note, the normal power-up sound will occur prior to this bleep.
- Release the Horn button, but continue to hold down the On/Off button until there are two further short bleeps.
- Release the On/Off button, there will now be a longer bleep and OBP mode will be entered. See previous diagram.




If the keycode method is set, then it may be possible for a wheelchair user to gain entry to the OBP function. PGDT accepts no liability for losses of any kind if this method of OBP is set. PGDT also accepts no liability for losses of any kind if OBP access is gained by unauthorised personnel obtaining a dongle.

# 2.2 Navigating the OBP Screens

Whilst in OBP deflecting the joystick to the left, whilst the parameter title is highlighted, will display an elongated version of the parameter name.

There are two basic forms of display screen – one which shows global parameters and one which shows parameters which are profile dependent, i.e. can have different values in each of the drive profiles.

Example of a global parameter screen:

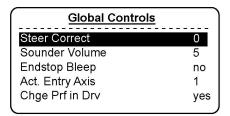

Forward joystick deflections will select the parameter above.

Reverse joystick deflections will select the parameter below.

Left joystick deflections will take you to the previous screen.

Right joystick deflections will have no effect.

Example of a profiled parameter screen:

FWD	Pr1	Pr2	Pr3	Pr4	
Acc ▲	30	35	40	45	
Acc ▼	20	20	20	20	
Dec▲	40	40	45	50	
Dec▼	30	30	30	30	
Spd▲	70	80	90	100	
Spd▼	20	25	30	20	

Forward joystick deflections will select the parameter above.

Reverse joystick deflections will select the parameter below.

The first left joystick deflection will display the parameter name in full.

The second left joystick deflection will take you to the previous screen.

Right joystick deflections will select the parameter to the right, if available.

# 2.3 Making an Adjustment

Once the parameter for adjustment has been selected, the Speed buttons can be used to make adjustments.



To ensure all new values are stored correctly, exit OBP mode with a series of left hand joystick deflections

# 3 PC Programmer - OEM Parameter List (\* Denotes present in OBP)

### Profile Management

Profile Name

Sets the Profile Name

\*Profile Fnable

Sets if a Profile is available

\*Mode Enable

Sets which Modes are available in specific Profiles

\*Input Device Type

Sets which Input Device is active within a Profile

Input Device Subtype

Sets which Input Device Subtype is active within a Profile

Seat Reversal Profile

Sets which Profiles relate to each configuration

Allow Grab

Sets which Input Device can grab focus.

Configuration

Mode Name Sets the Mode Name

Input Sets the data the Input Device transmits to the Output Device

Output Sets which Output Device is active for a specific Mode

Speeds

\*Maximum Forward Speed Sets maximum forward speed \*Minimum Forward Speed Sets minimum forward speed \*Maximum Reverse Speed Sets maximum reverse speed \*Minimum Reverse Speed Sets minimum reverse speed \*Maximum Turn Speed Sets maximum turnina speed \*Minimum Turn Speed Sets minimum turning speed \*Maximum Forward Acceleration Sets maximum forward acceleration \*Minimum Forward Acceleration Sets minimum forward acceleration \*Maximum Forward Deceleration Sets maximum forward deceleration \*Minimum Forward Deceleration Sets minimum forward deceleration \*Maximum Reverse Acceleration Sets maximum reverse acceleration \*Minimum Reverse Acceleration Sets minimum reverse acceleration \*Maximum Reverse Deceleration Sets maximum reverse deceleration \*Minimum Reverse Deceleration Sets minimum reverse deceleration \*Maximum Turn Acceleration Sets maximum turning acceleration \*Minimum Turn Acceleration Sets minimum turning acceleration \*Maximum Turn Deceleration Sets maximum turning deceleration

\*Minimum Turn Deceleration

\*Power Reduces power to minimize risk of indoor fittings damage

\*Torque boost to overcome obstacles at low speed settings

\*Tremor Damping Adjustable damping to reduce the effect of hand tremor

Fast Brake Rate Sets the Fast Brake Rate

#### Controls

Global Controls

\*Momentary Screens Enabled

Sets whether the momentary screens are enabled

\*Change Profile while Driving

Sets if it is possible to change between profiles whilst driving

\*Change Speed while Driving

Sets if the speed buttons are active whilst driving

Sets if the Joystick speed buttons are active

Profile Button

Sets the functionality of the profile button

\*Actuator Endstop Bleep Sets whether there is a bleep when a seat axis reaches endstop

\*Sounder Volume

Sets volume of audible feedback from JSM

\*Lock Function Enabled

Sets how the lock function is activated

\*Reverse Driving Alarm

Sets if the reverse driving alarm is active

\*Emergency Stop Switch Allows latched operation without the use of an Emergency Stop switch

OBP Keycode Entry

Sets the method of entry of OBP

\*Power-up Mode This sets the Mode that will be active when the system is powered-up.

Joystick

\*Joystick Forward Throw

Sets the amount of joystick movement required to reach full speed in this direction

\*Joystick Reverse Throw

Sets the amount of joystick movement required to reach full speed in this direction

\*Joystick Left Throw

Sets the amount of joystick movement required to reach full speed in this direction

\*Joystick Right Throw

Sets the amount of joystick movement required to reach full speed in this direction

\*Joystick Deadband (size of neutral position)

\*Invert Left Right JS Axis Sets if the joystick Left/Right axis is inverted

\*Invert Fwd Rev JS Axis Sets if the joystick Fwd/Rev axis is inverted

\*Swap Joystick Axis Swaps the joystick's Fwd/Rev and Left/Right axes

Profiled

\*Change Mode while Driving

Sets if it is possible to change between Modes whilst driving

\*Sleep Timer

Sets the period of inactivity before the Control System powers down

\*Standby Time

Sets the period of inactivity before Standby Mode is entered.

\*Switch to Standby

Sets if an external input switch can be used to enter Standby Mode

\*Mode Selection in Standby

Sets if the joystick can be used to select Modes when in Standby Mode

Standby in Modes Sets the Modes that Standby will operate in

Standby Forward

Sets which Mode is entered with a forward command

Standby Reverse

Sets which Mode is entered with a reverse command

Standby Left

Sets which Mode is entered with a left command

Standby Right

Sets which Mode is entered with a right command

\*Remote Selection Sets whether Profiles and Mode can be selected and entered with joystick deflections

\*Background Sets the default background for each profile

Latched

\*Latched Drive Selects latched drive operation

\*Latched Actuators Selects latched actuator operation

\*Latched Timeout Sets the timeout period for latched operation

\*Latched Timeout Bleep Sets whether a bleep occurs as the timeout period approaches

Seating

Seating module Enabled Sets which Module is active – SM or ISM

Intelligent Seating Module

Global

\*Actuator Mode Entry Axis Sets the default axis when Seating Mode is entered

Axis Setup

Axis Channels Sets which actuator motor channels on the ISM will operate for a particular axis

Axis Display Sets the user display for a particular axis

Invert Axis Direction

Sets whether a channel direction is inverted for a particular axis

Axis Name

Sets the text that will be displayed when an actuator is selected

Channels

\*Acceleration Sets the acceleration for each actuator channel
\*Deceleration Sets the deceleration for each actuator channel

\*Up Speed Sets the speed in the up direction for each actuator channel
\*Down Speed Sets the speed in the down direction for each actuator channel

Up Current Limit

Sets the maximum current output in the up direction for each actuator channel

Down Current Limit

Sets the maximum current output in the down direction for each actuator channel

Endstop Autodetect

Sets if the automatic endstop function for a particular actuator channel operates

Actuator Endstop Timeout Sets the period of time before the drive is cut at an endstop position

Seating Module Refer to chapter 4

Global

\*Actuator Mode Entry Axis Sets the default axis when Seating Mode is entered

Axis Setup

Axis Enabled Sets whether the actuator axis is enabled or not

\*Up Speed Sets the speed in the up direction for each actuator channel
\*Down Speed Sets the speed in the down direction for each actuator channel

Axis Endforce Sets the Endforce for a particular axis

Actuator Endstop Timeout

Sets the time after which endforce will not be applied to the actuator

Axis Display

Sets the text that will be displayed when an actuator is selected

Axis Inhibits

Up Assign

Sets whether the actuator axis is inhibited in the Up direction

Up: Inhibit in bands

Sets the bands in which the inhibit is active in the Up direction

Down Assign

Sets whether the actuator axis is inhibited in the Down direction

Down: Assign in bands

Sets the bands in which the inhibit is active in the Down direction

Battery

Low Battery Alarm Sets whether there will be an audible alarm to signal a low battery condition

Low Battery Flash Level Sets the point at which the battery gauge on the LCD screen starts to flash

Cable Resistance Calibrates the system to compensate for the electrical resistance of the battery cables

Calibration Factor Allows further fine calibration of the battery gauge Low Voltage Cut-out Sets the voltage level at which the low voltage timer will start

Low Voltage Time Sets the period of time that the voltage must be below the above parameter before drive is cut

General

Brake Disconnected Alarm Sets whether there is an audible alarm when a brake(s) is disconnected

Brake Fault Detect Sets whether the Power Module will detect electrical faults in the wheelchair's brakes Brake Voltage Sets the voltage output from the Power Module to the wheelchair's electrical brakes Soft Stop Rate Sets the deceleration rate that is used while the wheelchair is "soft-stopping"

Front Wheel Drive Selects specialist software for front wheel driven wheelchairs Seat Reversal Sets the various options for wheelchairs that have a reversing seat

Charger Inhibit 3 Enable Dedicates the Inhibit 3 input on the Power Module to operate with an On-Board Charger Charaer Inhibit 3 Latchina Sets whether the OBC inhibit (PM Inhibit 3) as described above is latching or non-latching

Charger Inhibit 1 Latchina Sets whether Inhibit 1 (JSM Charaer Socket) is latching or non-latching

Charger Inhibit 1 Alarm Sets whether an alarm sounds if there is an inhibit present on the JSM Charger Socket Charger Inhibit 3 Alarm Sets whether an alarm sounds if there is an inhibit present on the PM Inbibit 3

Motor Global

> Maximum Current Limit Sets the long term maximum current output of the Power Module **Boost Drive Current** Sets the short term maximum current output of the Power Module

Boost Drive Time Sets the period of time that the level of current set by Boost Drive Current is available Current Foldback Threshold Sets the level of current which, if exceeded, will start the current foldback timer

Current Foldback Time Sets the period of time that the current must exceed the above parameter before the current is reduced

Current Foldback Level Sets the level of current output reduction

Compensation Matches the PM to the motors

Invert M1 Direction Inverts the direction of rotation of motor channel M1 on the Power Module Invert M2 Direction Inverts the direction of rotation of motor channel M2 on the Power Module Motor Swap Swaps the motor output channels, M1 and M2, of the Power Module \*Steer Correct Adjusts the PM outputs to compensate for mis-matched motors

\*Display Speed Sets whether the speed and odometer is displayed in miles per hour or kilometers per hour

Maximum Rated Speed This calibrates the speedometer function

\*Maximum Displayed Speed This sets the operation of the graphical speed display

Profiled

Allows different values of Compensation to be used across different profiles. Compensation Factor

Inhibits Inputs - PM

Lower Level Threshold

Sets the threshold between Band O and Band 1 for a particular Inhibit Input on the PM Middle Level Threshold Sets the threshold between Band 1 and Band 2 for a particular Inhibit Input on the PM Upper Level Threshold Sets the threshold between Band 2 and Band 3 for a particular Inhibit Input on the PM Sets a debounce time for the inhibit input.

Debounce Time

Inputs - ISM

Lower Level Threshold Sets the threshold between Band 0 and Band 1 for a particular Inhibit Input on the ISM Middle Level Threshold Sets the threshold between Band 1 and Band 2 for a particular Inhibit Input on the ISM Upper Level Threshold Sets the threshold between Band 2 and Band 3 for a particular Inhibit Input on the ISM

Debounce Time Sets a debounce time for the inhibit input

Drive Inhibits

Links the software inhibits to an actual hardware inhibit

Speed Limit in Band 0 Sets the max speed when the Inhibit Input, which is assigned to the Drive Inhibit, is in Band 0 Speed Limit in Band 1 Sets the max speed when the Inhibit Input, which is assigned to the Drive Inhibit, is in Band 1 Speed Limit in Band 2 Sets the max speed when the Inhibit Input, which is assigned to the Drive Inhibit, is in Band 2Speed Limit in Band 3 Sets the max speed when the Inhibit Input, which is assigned to the Drive Inhibit, is in Band 3

Operation Sets whether a full drive inhibit is a latching or non-latching function Inhibit Alarm Sets whether an audible alarm will sound in a full drive inhibit condition

Actuator Channel 'X' Inhibits

Up: Assign Links the software inhibits to an actual hardware inhibit

Up: Inhibit in Bands Sets the condition of the input that will result in an inhibit of an actuator channel in the up direction Up: Alarm Sets whether an audible alarm will sound when an actuator channel is inhibited in the up direction

SK77981/5 67 Down: Assign Links the software inhibits to an actual hardware inhibit

Down: Inhibit in Bands

Sets the condition of the input that will result in an inhibit of an actuator channel in the down direction

Down: Alarm

Sets whether an audible alarm will sound when an actuator channel is inhibited in the down direction

Lights

Lamp Voltage Sets the operating voltage for the wheelchair's lighting system Indicator Fault Detection Sets whether defective turn indicator lamps can be detected

Swap Indicators Seat Reversal

Sets whether the indicator channels are exchanged when the seat is reversed

Brake Lights or Horn

Sets whether the spare outputs on the ISM are assigned as Brake Lights or Horn

**OEM Factory** 

Absolute Max Torque Sets the maximum programmable value Absolute Minimum Power Sets the minimum programmable value Absolute Maximum Forward Speed Sets the maximum programmable value Absolute Minimum Forward Speed Sets the minimum programmable value Absolute Maximum Reverse Speed Sets the maximum programmable value Absolute Minimum Reverse Speed Sets the minimum programmable value Absolute Maximum Turning Speed Sets the maximum programmable value Absolute Minimum Turning Speed Sets the minimum programmable value Absolute Maximum Forward Acceleration Sets the maximum programmable value Absolute Minimum Forward Acceleration Sets the minimum programmable value Absolute Maximum Forward Deceleration Sets the maximum programmable value Absolute Minimum Forward Deceleration Sets the minimum programmable value Absolute Maximum Turn Acceleration Sets the maximum programmable value Absolute Minimum Turn Acceleration Sets the minimum programmable value Absolute Maximum Turn Deceleration Sets the maximum programmable value Absolute Minimum Turn Deceleration Sets the minimum programmable value Absolute Maximum Reverse Acceleration Sets the maximum programmable value Absolute Minimum Reverse Acceleration Sets the minimum programmable value Absolute Maximum Reverse Deceleration Sets the maximum programmable value Absolute Minimum Reverse Deceleration Sets the minimum programmable value Latched Drive Enabled Disables latched drive

Encoders Refer to Chapter 5

Encoder Module fitted

Sets whether R-net checks for the presence of an Encoder Module

Encoder Error Speed Limit

Sets the maximum speed should an Encoder error be detected

Invert M1 Inverts the signals being read from the Encoder connector M1 of the Power Module
Invert M2 Inverts the signals being read from the Encoder connector M2 of the Power Module

Encoder Resolution Sets the number of pulses per revolution that the encoders output.

Motor Rated Speed Sets the Motor Rated Speed at 24V.

Mouse

Forward Nudge

Sets the action when the JS is deflected forward in mouse mode

Reverse Nudge

Sets the action when the JS is deflected in reverse in mouse mode

Left Nudge

Sets the action when the JS is deflected left in mouse mode

Right Nudge

Sets the action when the JS is deflected right in mouse mode

Nudge Time

Sets the time for a nudge to be active

Action Beeps

Enables beeps on clicks on scroll functions

Deflection beeps Enables beeps on deflection outside the deadband

# 4 Profile Management

### :: Profiles And Modes Explained ::

The operation of the R-net system is based around the concept of Profiles and Modes.

A Profile is a collection of programmable parameters that affect the operation and performance of the wheelchair. It is typical to have 5 Profiles, each set-up to give a different driving performance. For example, Profile 1 may have very slow settings for speeds and accelerations, while Profiles 2 through to 5 would have progressively faster settings. In this way, the user can change the performance of the wheelchair by selecting the appropriate Profile with the Profile Button on the JSM. Although Profiles are used mainly to change driving performance, they can also contain parameters for other functions, such as seating control.


A Mode reflects the actual operation that occurs as a result of a joystick command. Typical examples are Drive Mode, Seating Control Mode and Bluetooth Mode. The user can select the required Mode with the Mode Button on the JSM. Note, it will only be possible to select a Mode if there is a Module that will support that Mode connected into the system. For example, if there is no Mouse Module connected, then the user will not be able to select that Mode.

#### : Profile Management :


The R-net system can have up to 8 Profiles. Conventionally, Profile 8 is reserved for Dual Attendant Control (but can be reassigned), meaning there are 7 possible user Profiles; but normally just 5 are used. The parameter, Profile Enable, sets whether a particular Profile is available to the user.

Operations of the Profile Button on the JSM simply select the next available Profile. When the highest available Profile is reached, the next Profile Button operation will simply select the lowest available Profile.

The LCD on the JSM displays the currently selected Profile as a text string of up to 20 characters. The number of the Profile is also displayed.

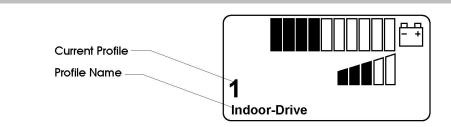

The Profile naming text is programmable, meaning you can choose your own description and language. The parameter, Profile Name, sets the text. Below is an example of programming for both Profile availability and naming.

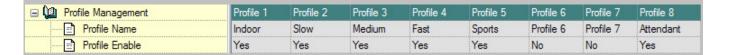


Profiles 1 to 5 are available to the user and have been given meaningful names. Profiles 6 and 7 are not available, and Profile 8 has been assigned to Dual Attendant operation.

# : Mode Management :

Within each available Profile, it is possible to select 7 different operating Modes. There are actually 8 Modes, however, Mode 8 is always reserved for On-Board Programming (OBP). A standard wheelchair is likely to have just 2 operating Modes – Drive (Mode 1) and Seating Control (Mode 2) – however, it is normal to set all Modes to be available, so that they will automatically be

accessible when an additional Output Module is connected. If they are not programmed to be available, then if an additional Output Module, such as a Bluetooth Mouse Module is connected, the system would have to be programmed to recognize this new Module.

The parameter that sets whether a Mode is available is Mode Enable. Below is an example of programming to satisfy this requirement.

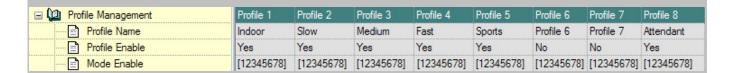

However, this programming is not appropriate if it may be required to use just one button or switch to select Profiles <u>and</u> Modes. For this reason, it is recommended that programming as described in the following section is used.

# : Selecting Profile and Modes with Just One Button :

It may sometimes be required to use just one button or switch to select Profiles <u>and</u> Modes. A typical example would be the use of a Buddy Button connected into the External Profile Switch jack socket. The R-net system has been designed to allow this by applying the following rule.

Each operation of the External Profile Switch will select the next available Profile. Upon reaching the highest available Profile, the next operation of the switch will select the next available Mode, regardless of whether the Mode is available in the current Profile. PGDT therefore recommend a programming set-up as below.

☐ Profile Management	Profile 1	Profile 2		Profile 3		Profile 4		Profile 5		Profile 6	Profile 7		Profile 8
Profile Name	Indoor	Slow		Medium		Fast		Sports		Profile 6	Profile 7		Attendant
Profile Enable	Yes	Yes		Yes		Yes		Yes		No	No		Yes
Mode Enable	[12345678]	[1	]	[1	]	[1	]	[1	]	[1 ]	[1	]	[12345678]

In this set-up the External Profile Switch would step through Drive Profiles 1 to 5, then to Mode 2 (Seating Control) and then to other Modes if appropriate Output Modules are connected.

Note, if the programming set-up in the previous section was used, then the External Profile Switch would step through Drive Profiles 1 to 5, then it would step through 5 Seating Control Modes, each effectively the same. If more Output Modules were connected, it would be necessary to step through 5 times for each, before getting back to Drive Profile 1. Likewise, 5 steps through OBP would be needed to exit that function.

Note, the programming in this section will also work in the correct fashion when separate Profile and Mode buttons are used.

The basic rules are:

The Profile Button will select the next available Profile in the current Mode.

The Mode Button will select the next available Mode in the last Profile in which that Mode was used.

The External Profile Jack will select the same Mode in the next available Profile. When the highest available Profile for the current Mode is reached, then the next Mode in the next available Profile will be selected.

#### 4.I Profile Name

This is a 20 character text string that will appear on the JSM LCD screen to indicate the Profile that is selected. More information is given in the above text.

### 4.2 Profile Enable - (PrfEn)

This sets whether a particular Profile is available for selection by the wheelchair user. More information is given in the above text.




# A Control System utilizing an LED JSM should only be programmed with a maximum of 5 Profiles enabled.

### 4.3 Mode Enable (Modes)

This sets which operating Modes are available in each of the Profiles. More information is given in the above text.

If this parameter is selected, a drop down menu as below will appear.

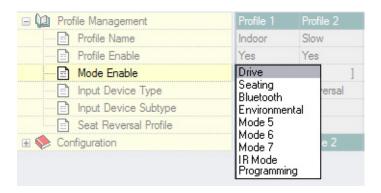

To set whether a Mode is enabled, simply highlight the appropriate menu item. Any combination of Modes can be selected.

When the drop down menu is closed the selected Modes are expressed as numbers and the parameter appears on the screen as below. The numbers 1 to 8 refer to the Modes listed in the menu.



#### :: Input Devices Explained ::

It is possible to connect two Input Devices of different types, for example, a conventional JSM and a Head Control into the same system. Alternatively, it is sometimes (but rarely) required, to connect two Input Devices of the same type.

The parameters Input Device Type, Input Device Subtype and Allow Grab, allow you to manage the priorities of control of the two connected Input Devices.

If two Input Devices are connected, the Profile buttons on those Input Devices can select only the Profiles relevant to themselves. For example, if Input Device 1 is set for Profiles 1, 2 and 3, and Input Device 2 is set for Profiles 4, 5 and 6; then the Profile button on Input Device 1 will be able to select just Profiles 1, 2 and 3, and the Profile Button on Input Device 2 will be able to select just Profiles 4, 5 and 6.

# 4.4 Input Device Type - (ID Type)

Each Profile can be set to work with any type of Input Device, or only with a specific type of Input Device. The parameter, Input Device Type, is used to set this function and there are currently twelve settings – Universal, JSM, Attendant, Omni, Switch-it, IOM 1, IOM 2, IOM 3, Compact JS, Compact JS Advanced, Easy Rider and Magic Drive.

If set to Universal, then any Input Device can control the Profile, and it will be the Input Device that was used to powerup the system that will be in control.

SK7798I/5 7I

If set to anything other than Universal, then only the programmed type of Input Device can control the Profile.

#### 4.5 Input Device Subtype

In the rare event of it being required to have two identical Input Devices connected into the same system, for example, a left and right hand JSM, then the parameter, Input Device Subtype, can be used to manage this situation. There are three settings – All, 1 and 2. Note, this parameter is only relevant if the Input Devices are identical. In all other instances, it should be ignored.

If set to All, then either Input Device can control the Profile, and it will be the Input Device used to power-up the system that will be in control.

If set to 1, then only the older of the two Input Devices will be able to control the Profile.

If set to 2, then only the younger of the two Input Devices will be able to control the Profile.

The relative ages of the two Input Devices will be determined automatically by the R-net system. However, it will also be necessary for the Input Devices to be manually aged, in order to identify which is actually which. The relative ages are determined by the serial numbers, which have a format as below.

#### AXYYMM1234

YY and MM relate to year and month of manufacture. If there is no differentiation using this method, then the last four series digits will be used.

#### 4.6 Seat Reversal Profile

This parameter is only relevant to wheelchair models that have a reversing seat, i.e. wheelchairs that can be operated in both rear wheel drive and front wheel drive configuration. The purpose of the parameter is to set which Profiles relate to each configuration, as it is normal to have different drive characteristics for the two set-ups.

There are two programmable options yes and no.

If set to yes, then the Profile will only be available when the seat is reversed. The seat is considered to be reversed when the Inhibit 2 input on the Power Module is at 0V. For more details on seat reversal, please refer to section, Seat Reversal.


If set to no, then the Profile will only be available when the seat is not reversed.

#### 4.7 Allow Grab

This parameter can be used to prevent one type of Input Device conceding control to another.

Allow Grab determines if the Input Device can concede focus or not.

If this parameter is selected a dialog box as below will appear.

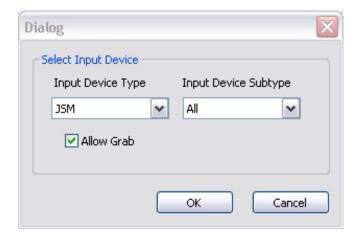

Example 1: To prevent a user gaining control from the Attendant.

The system is set up as below.



Profiles 1 to 5 are set for any type of user Input Device

Profile 8 is set for Attendant control

Allow Grab is set to Yes for Profiles 1 to 5 meaning the Attendant can grab focus from any Input Device connected.

Allow Grab is set to No for Profile 8 meaning that other Input Devices cannot grab focus from the Attendant.

Example 2: In a playground environment a user must have the ability to take focus from the Attendant Module. The system is set up as below.



Profiles 1 to 5 are set for any type of user Input Device

Profile 8 is set for Attendant control

Allow Grab is set to Yes for Profiles 1 to 5 meaning the Attendant can grab focus from any Input Device connected.

Allow Grab is set to Yes for Profile 8 meaning that other Input Devices can grab focus from the Attendant.



It is not possible to set Allow Grab if the Input Device Type is Universal or IOM I - 3.



It is not possible to set Allow Grab to No on more than one Input Device Type.

# 5 Configuration


## :: Configuration Explained ::

The R-net system supports 8 Modes of operation. A Mode is the output action that the system is undertaking, for example, driving, seat adjustment or Mouse control. In other words, each Mode requires a piece of output hardware. For Drive it would be the Power Module, for Seat adjustment it would be the Seating Module and for Mouse control it would be a Bluetooth Mouse Module. Note, On-board Programming (OBP) will take over a Mode and by convention this will be Mode 8.

The system can be programmed to set which output function is active for each mode. In general, Mode 1 will always be Drive, Mode 2 will always be Seating Control and Mode 8 will always be OBP. However, it may be desirable to change the output assignation of some of the more specialist Modes. This can achieved with the parameter, Output. See below for more details.

A Mode will only be available if the output hardware is actually connected in the system. The parameter, Output, is therefore essential in ensuring the full "Plug & Play" capabilities of R-net are utilized.

Example: A wheelchair model is launched with Drive and Seating functions, but it is intended to allow optional features such as environmental control or PC mouse control to be fitted in the field. To ensure this means no more than simply connecting appropriate pieces of hardware, then the system should be programmed as below.

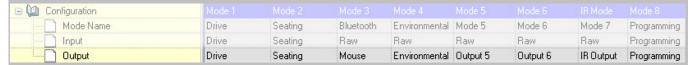

This means that when the wheelchair is initially shipped, with a Power Module and Seating Module fitted, then Modes 1 and 2 will be Drive and Seating Control respectively and that Mode 8 would be used for OBP.

If, at some point in the future an Environmental Control Module is required, then Mode 4 would automatically become available to the user. I.e. the Mode button would step through Drive, Seating and Environmental Modes. The other Modes - PC Mouse, Output 5, Output 6 and Output 7 - would also become available if relevant modules were connected.

Within each Mode it is necessary to ensure the appropriate data is transmitted from the Input Device to the Output Device. A good example of this is the Speed Up and Speed Down button. In Drive Mode they change the speed setting in the Joystick Module and it is that setting, rather than the actual state of the buttons, that is transmitted to the Power Module. However, in Mouse Mode it is desirable for the state of the Speed Up and Speed Down buttons to be transmitted on the Output Device. This means the buttons can be used to control other functions, such as mouse left/right clicks. The parameter, Input, sets the data for each mode. See section, Input, for more details.

## : User Displays :

There is some flexibility as to what is displayed to the user to indicate the operating Mode of the system. The Drive, Seating and OBP modes are indicated by the standard screens and cannot be changed. However, the Mouse Mode displays an image of a PC with an associated text string, whereas the other Modes simply have an associated text string. An example of a Mode screen is shown below. The text string is programmable via the parameter, Mode Name.



**Bluetooth Mode** 

# 5.I Mode Name

This is a 20 character text string that will appear on the JSM LCD screen to indicate the active Mode. See :: Configuration Explained :: for more details.




Although there are options to change the Mode Names for Drive and Seating Modes, these text strings will not appear on the standard JSM screens. These text options are reserved for future Input Devices.

## 5.2 Input

This sets the data that the Input Device transmits to the Output Device for each mode. See :: Configuration Explained :: for more details

The programmable options are in the form of a drop-down menu, as shown in the following diagram.

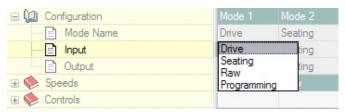

Drive provides data that is appropriate for driving the wheelchair and should always be used for Drive mode, i.e. when the Power Module is the active Output Device.

Seating provides data that is appropriate for controlling the seating of the wheelchair and should always be used for Seating Control mode, i.e. when an ISM, or equivalent, is the active Output Device.


Raw provides basic joystick x and y data, but scaled for deadband, throw and orientation. Additionally, 'raw' button operations are also transmitted. This setting can be used for any other Output Module types.

Programming sets which Mode is reserved for OBP. This is usually Mode 8.

## 5.3 Output

This sets the Output Device that is active for the Mode. See :: Configuration Explained :: for more details.

The programmable options are in the form of a drop-down menu, as shown in the following diagram

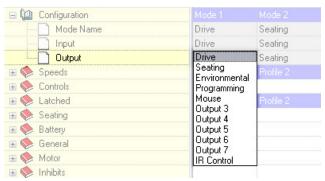

Drive means that the Power Module is the active Output Device and Seating means that the Seating Module is the active Output Device.

Because OBP assumes a Mode, then if OBP is required one of the Modes, normally Mode 8, should be set to Programming.

There are some other Output Device identifiers that a currently defined, such as Environmental, these are for use with future planned Output Modules.

The settings Output 3 to Output 7 are unique identifiers that can be assigned to Input/Output Modules (IOMs).



The number of unique identifiers is unlimited meaning any number of specialist modules can be supported.

# 6 Speeds

## 6.I Maximum Forward Speed - (Fwd Speed Max)

Sets the forward driving speed of the wheelchair when the joystick is deflected full ahead and the speed setting is at maximum (all 5 bars illuminated).

Programmable range is 0 to 100% in steps of 1%.

#### 6.2 Minimum Forward Speed - (Fwd Speed Min)

Sets the forward driving speed of the wheelchair when the joystick is deflected full ahead and the speed setting is at minimum (just 1 bar illuminated).

Programmable range is 0 to Maximum Forward Speed in steps of 1%.



It is not possible to set the value of Minimum Forward Speed higher then the value of Maximum Forward Speed.

## 6.3 Maximum Reverse Speed - (Rev Speed Max)

Sets the reverse driving speed of the wheelchair when the joystick is deflected to full reverse and the speed setting is at maximum (all 5 bars illuminated).

Programmable range is 0 to 100% in steps of 1%.

## 6.4 Minimum Reverse Speed - (Rev Speed Min)

Sets the reverse driving speed of the wheelchair when the joystick is deflected to full reverse and the speed setting is at minimum (just 1 bar illuminated).

Programmable range is 0 to Maximum Reverse Speed in steps of 1%.



It is not possible to set the value of Minimum Reverse Speed higher then the value of Maximum Reverse Speed.

## 6.5 Maximum Turning Speed - (Turn Speed Max)

Sets the turning speed of the wheelchair when the joystick is deflected fully left or right and the speed setting is at maximum (all 5 bars illuminated).

Programmable range is 0 to 100% in steps of 1%.

## 6.6 Minimum Turning Speed - (Turn Speed Min)

Sets the turning speed of the wheelchair when the joystick is deflected full left or right and the speed setting is at minimum (just 1 bar illuminated).

Programmable range is 0 to Maximum Turning Speed in steps of 1%.



It is not possible to set the value of Minimum Turning Speed higher then the value of Maximum Turning Speed.

#### 6.7 Maximum Forward Acceleration - (Fwd Accel Max)

Sets the acceleration rate of the wheelchair in the forward direction when the speed setting is at maximum (all 5 bars illuminated).

Programmable range is 0 to 100% in steps of 1%.

#### 6.8 Minimum Forward Acceleration - (Fwd Accel Min)

Sets the acceleration rate of the wheelchair in the forward direction when the speed setting is at minimum (just 1 bar illuminated). Programmable range is 0 to Maximum Forward Acceleration in steps of 1%.



It is not possible to set the value of Minimum Forward Acceleration higher then the value of Maximum Forward Acceleration.

#### 6.9 Maximum Forward Deceleration - (Fwd Decel Max)

Sets the deceleration rate of the wheelchair in the forward direction when the speed setting is at maximum (all 5 bars illuminated).

Programmable range is 0 to 100% in steps of 1%.

#### 6.10 Minimum Forward Deceleration - (Fwd Decel Min)

Sets the deceleration rate of the wheelchair in the forward direction when the speed setting is at minimum (just 1 bar illuminated). Programmable range is 0 to Maximum Forward Deceleration in steps of 1%.



It is not possible to set the value of Minimum Forward Deceleration higher then the value of Maximum Forward Deceleration.

## 6.II Maximum Reverse Acceleration - (Rev Accel Max)

Sets the acceleration rate of the wheelchair in the reverse direction when the speed setting is at maximum (all 5 bars illuminated).

Programmable range is 0 to 100% in steps of 1%.

#### 6.12 Minimum Reverse Acceleration - (Rev Accel Min)

Sets the acceleration rate of the wheelchair in the reverse direction when the speed setting is at minimum (just 1 bar illuminated). Programmable range is 0 to Maximum Reverse Acceleration in steps of 1%.



It is not possible to set the value of Minimum Reverse Acceleration higher then the value of Maximum Reverse Acceleration.

## 6.13 Maximum Reverse Deceleration - (Rev Decel Max)

Sets the deceleration rate of the wheelchair in the reverse direction when the speed setting is at maximum (all 5 bars illuminated).

Programmable range is 0 to 100% in steps of 1%.

## 6.14 Minimum Reverse Deceleration - (Rev Decel Min)

Sets the deceleration rate of the wheelchair in the reverse direction when the speed setting is at minimum (just 1 bar illuminated).

Programmable range is 0 to Maximum Reverse Deceleration in steps of 1%.



It is not possible to set the value of Minimum Reverse Deceleration higher then the value of Maximum Reverse Deceleration.

#### 6.15 Maximum Turn Acceleration - (Turn Accel Max)

Sets the acceleration rate of the wheelchair into a turn when the speed setting is at maximum (all 5 bars illuminated). Programmable range is 0 to 100% in steps of 1%.

#### 6.16 Minimum Turn Acceleration - (Turn Accel Min)

Sets the acceleration rate of the wheelchair into a turn when the speed setting is at minimum (just 1 bar illuminated). Programmable range is 0 to Maximum Turn Acceleration in steps of 1%.



It is not possible to set the value of Minimum Turn Acceleration higher then the value of Maximum Turn Acceleration.

#### 6.17 Maximum Turn Deceleration - (Turn Decel Max)

Sets the deceleration rate of the wheelchair while turning when the speed setting is at maximum (all 5 bars illuminated). Programmable range is 0 to 100% in steps of 1%.

#### 6.18 Minimum Turn Deceleration - (Turn Decel Min)

Sets the deceleration rate of the wheelchair while turning when the speed setting is at minimum (just 1 bar illuminated). Programmable range is 0 to Maximum Turn Deceleration in steps of 1%.



It is not possible to set the value of Minimum Turn Deceleration higher then the value of Maximum Turn Deceleration.

## 6.19 Power - (Power)

This reduces the power of the wheelchair. Power is the ability of a wheelchair to climb a hill or overcome an obstacle. If it is set to 100% then the wheelchair will provide full power. Values below 100% will result in reduced power.

A typical use is minimizing damage to doorways or furniture if the wheelchair is being used indoors. The values can be set independently between drive profiles, meaning separate indoor and outdoor profiles can be defined.

Programmable range is 25% to 100% in steps of 1%.

If the value is set to less than 100%, then the Boost Current function is automatically disabled. i.e. the reduction is applied to the Current Limit Max. value.

Example: R-net is programmed to: Maximum Current Limit = 80 Amps

Boost Drive Current = 120 Amps
Boost Drive Time = 5 Seconds
Power (Profile 1) = 100%
Power (Profile 2) = 50%

This means that in Profile 1 the system will be able to output 120A for 5 seconds before reducing to 80A, but in Profile 2 will output 50% of 80A = 40A with no boost.

#### 6.20 Torque - (Torq)

This parameter can be used to boost the power to the motors at low drive speeds. This is useful for overcoming obstacles such as door thresholds or thick pile carpets and for countering Rollback.

Programmable range is 0% to 100% in steps of 1%.

At 0% the parameter has no effect. The recommended value is 80%, however, if the wheelchair drive is considered to be jerky, then reduce this value.

## 6.21 Tremor Damping - (Trmor)

This parameter can be used to reduce the effects of a user's hand tremor.

The programmable range is 0% to 100%. The higher the value, the greater the damping that is applied



If using high values of Tremor Damping, pay particular attention to the stopping distance of the wheelchair, as it will be increased.

#### 6.22 Fast Brake Rate

This parameter sets the deceleration rate that is used while fast braking. Fast braking is when the joystick is pulled to the reverse position to make a faster stop.

The programmable range is 0 to 200.

If the value is set lower than the Forward Deceleration value, then the latter value will be used.

## 7 Controls

# Controls - Global

## 7.1 Momentary Screens Enabled - (Momentary Screens)

This sets whether the large, momentary screens appear in between Profile and Speed setting changes.

The programmable options are Yes and No.

## 7.2 Change Profile while Driving - (Chge Prf in Drv)

This sets whether Profile changes are permitted while the wheelchair is driving.

The programmable options are Yes and No.

# 7.3 Change Speed while Driving - (Spd Adj in Drv)

This sets whether the Speed Down and Speed Up buttons are active while the wheelchair is driving.

The programmable options are Yes and No.

## 7.4 Speed Adjust - (Speed Adjust)

This sets whether the Speed Down and Speed Up buttons on the Joystick Module are active.

The programmable options are Yes and No.

#### 7.5 Profile Button

This sets whether the Profile button on the JSM will select just Profiles, or Profiles and Modes.

The programmable options are Profiles and Profiles/Modes.

If set to Profiles, then the Profile button will select just the available Profiles.

If set to Profiles/Modes, then the Profile button on the JSM will select the available Profiles, then the available Modes. i.e. the operation is identical to a button connected into the External Profile Switch Jack.

## 7.6 Actuator Endstop Beep - (Endstop Beep)

This sets whether a short beep occurs when an actuator mechanism reaches an end position.

The programmable options are Yes and No.



The beep will only occur on actuator mechanisms that use automatic endstop detection, i.e. overcurrent measurement.

## 7.7 Sounder Volume - (Sounder Volume)

This sets the volume of the audible feedback given whenever a button or switch on the Joystick Module is operated.

The programmable range is 0 to 10 in steps of 1.

If set to 0, there will be no sound given. Higher values will give progressively louder sounds.

#### 7.8 Lock Function Enabled - (Lock Fn Enable)

This sets if and how the R-net may be locked to prevent the wheelchair being used by unauthorised persons.

The programmable options are as follows:

None Means the R-net cannot be locked

Seq Means the keypad locking sequence can lock/unlock the R-net.

(Refer to Chapter 1 - Operation for sequence details).

Key Means a PGDT supplied key can lock/unlock the R-net.

Both Means either the keypad or a key can be used to lock/unlock the R-net.

# 7.9 Reverse Driving Alarm - (Rev Driving Alarm)

This sets whether a warning alarm is sounded while the wheelchair is driving in reverse.

The programmable options are Yes and No.

## 7.10 Emergency Stop Switch - (Emergency Stop)

This sets whether an Emergency Stop Switch must be used when latched operation is programmed.

The programmable options are Yes and No.

If set to Yes, then a normally closed Emergency Stop Switch must be connected into the External Profile Switch Jack on the Joystick Module, whenever latched drive or latched actuator operation is programmed. If such a switch is not fitted, then operation of the wheelchair will be inhibited.

If set to No, then an Emergency Stop Switch is not required. This setting should only be used after a full assessment of the risks involved, i.e. there is no way of effecting an emergency stop from latched drive or actuator operation.



PGDT accepts no liability for losses of any kind resulting from the use of a wheelchair with latched drive or actuator operation and with no normally closed Emergency Stop Switch fitted.

## 7.II OBP Keycode Entry

This sets the method of entry into On-Board Programming (OBP).

The programmable options are Yes and No.

If set to Yes, then a sequence of button presses can be used to enter OBP. The sequence is as detailed at the end of this section.

If set to No, then a hardware Dongle must be connected into the system in order to enter OBP.

OBP keycode entry is as follows:

With the system switched off, hold down the Horn button and then hold down the On/Off button until there is a short bleep. Note, the normal power-up sound will occur prior to this bleep.

Release the Horn button, but continue to hold down the On/Off button until there are two further short bleeps.

Release the On/Off button, there will now be a longer bleep and OBP mode will be entered.




If this parameter is set to Yes, then it may be possible for a wheelchair user to gain entry to the OBP function. PGDT accepts no liability for losses of any kind if this method of OBP is set. PGDT also accept no liability for losses of any kind if OBP access is gained by unauthorised personnel obtaining a dongle.

## 7.12 Power-up Mode - (Power Up Mode)

This sets the Mode that is selected when the system is powered-up.

The programmable options are in the form of a drop down, as shown in the following diagram.

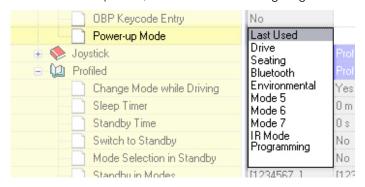

Simply select the required Mode. If Last Used is selected, then the Mode that was selected at the last power-down will be selected at the next power-up.

If a Mode is selected but the appropriate Output Module is not present, then the system will default to Drive Mode at power-up.



This parameter is not applicable to IOMs

SK7798I/5 8I

#### Controls - Joystick

#### 7.I3 Joystick Forward Throw - (Forward Throw)

This sets the amount of forward movement of the joystick that is required to reach full forward speed. This is particularly useful for users with limited hand movement.

The programmable range is 25% to 100% in steps of 1%.

The percentage value corresponds to the amount of joystick deflection required to reach full forward speed. For example, if set to 50%, then only half joystick movement is need to reach full speed.

The Forward Throw can also be set interactively with the user actually deflecting the joystick. See :: Interactive Setting Explained ::.

## 7.14 Joystick Reverse Throw - (Reverse Throw)

This sets the amount of reverse movement of the joystick that is required to reach full reverse speed. This is particularly useful for users with limited hand movement.

The programmable range is 25% to 100% in steps of 1%.

The percentage value corresponds to the amount of joystick deflection required to reach full reverse speed. For example, if set to 50%, then only half joystick movement is need to reach full speed.

The Reverse Throw can also be set interactively with the user actually deflecting the joystick. See :: Interactive Setting Explained ::..

#### 7.15 Joystick Left Throw - (Left Throw)

This sets the amount of left movement of the joystick that is required to reach full left turning speed. This is particularly useful for users with limited hand movement.

The programmable range is 25% to 100% in steps of 1%.

The percentage value corresponds to the amount of joystick deflection required to reach full left turning speed. For example, if set to 50%, then only half joystick movement is need to reach full speed.

The Left Throw can also be set interactively with the user actually deflecting the joystick. See :: Interactive Setting Explained ::.

## 7.16 Joystick Right Throw - (Right Throw)

This sets the amount of right movement of the joystick that is required to reach full right turning speed. This is particularly useful for users with limited hand movement.

The programmable range is 25% to 100% in steps of 1%.

The percentage value corresponds to the amount of joystick deflection required to reach full right turning speed. For example, if set to 50%, then only half joystick movement is need to reach full speed.

The Right Throw can also be set interactively with the user actually deflecting the joystick. See :: Interactive Setting Explained ::.

# :: Interactive Setting explained ::

Setting the joystick throw interactively can only be achieved using OBP or a DTT.

To navigate and make selections in OBP use the joystick, deflecting it left, right, up and down as required. When a menu item is highlighted deflect the joystick to the right to enter.

If an item name is abbreviated then deflecting the joystick to the left will display its full name. A second deflection to the left will exit the current screen.

Once a parameter has been selected using the Speed Up and Speed Down buttons on the JSM's keypad will effect changes, unless an on screen message details otherwise.

To set Throw Interactively using OBP use the following procedure:

- Enter OBP and select Controls.
- From the Controls Menu select Joystick.

- From the Joystick Menu select Active Throw.
- From the Throw Menu set which Profiles will be affected by the interactive setting. This is achieved by placing a tick
  next to the Profiles required and a cross by the Profiles not required. Use the Speed Up and Down buttons to
  change the setting.
- Once this is complete deflect the joystick to the right while any of the Profiles are Highlighted. This will enter the Interactive Throw programming procedure.
- Follow the on-screen instructions
- After the Right direction has been set the R-net will go through a configuration process, an hourglass will be displayed during this time.
- Once the configuration is complete a large tick will be displayed on the screen for a short time and then the
  Joystick Menu will be displayed.

## 7.17 Joystick Deadband - (Deadband)

This sets the size of the joystick's neutral area. In other words, how far the joystick has to be deflected before the brakes are released and drive commences.

The programmable range is 10% to 50% in steps of 1%. A normal setting is 10%.

# 7.18 Invert Fwd Rev JS Axis , Invert Left Right JS Axis & Swap Joystick Axis - (Invert F/R, Invert L/R & Swap Axes)

These three parameters are closely related, so are covered in one section. The parameters allow eight different joystick direction configurations, or orientations. For example, it may be required that the joystick has to be pulled in reverse to effect forward drive. In this instance, it would simply be a case of setting Invert Fwd Rev JS Axis to Yes, however, there are many other combinations and these are best explained in the form of a table.

Required Orientation			Required Programming			
Forward	Reverse	Left	Right	Invert F/R	Invert L/R	Swap Axis
Forward	Reverse	Left	Right	No	No	No
Reverse	Forward	Left	Right	Yes	No	No
Forward	Reverse	Right	Left	No	Yes	No
Reverse	Forward	Right	Left	Yes	Yes	No
Left	Right	Reverse	Forward	No	No	Yes
Left	Right	Forward	Reverse	Yes	No	Yes
Right	Left	Reverse	Forward	No	Yes	Yes
Right	Left	Forward	Reverse	Yes	Yes	Yes

## Controls - Profiled

## 7.19 Change Mode while Driving - (CMid)

This sets whether Mode changes are permitted while the wheelchair is driving.

The programmable options are Yes and No.

## 7.20 Sleep Timer - (Sleep)

This sets the period of joystick (or other type of Input Device) inactivity before the system will automatically power down.

The programmable range is 0 minutes to 30 minutes in steps of 1 minute. If set to a value of 0, then the system will never automatically power-down.

#### :: Standby Operation Explained ::

Standby Mode provides a means for the user to select Profile and Modes without using switches or buttons. This is achieved by using joystick deflections to select and enter Profiles and/or Modes.

There are two ways to enter Standby Mode. These are set by the parameters, Standby Time and Switch to Standby, and are explained below.

While in Standby Mode, the operation of the joystick is set by the parameters, Mode Selection in Standby, Standby Forward, Standby Reverse, Standby Left, Standby Right and Remote Selection. Each is explained below.

## 7.21 Standby Time - (Sbytm)

This sets a period of time of joystick inactivity before Standby Mode is entered.

The programmable range is 0 to 200 seconds. A value of 0 means that Standby Mode will never be entered.

## 7.22 Switch to Standby - (SwSby)

This sets whether a switch connected into the External Profile Switch Jack can be used to enter Standby Mode.

The programmable options are Yes and No.

If set to Yes, an external switch can be used to enter Standby Mode.

If set to No, Standby Mode cannot be entered with an external switch.

#### 7.23 Mode Selection in Standby - (MdeSI)

This sets whether the joystick can be used to enter different operating Modes while the system is in Standby Mode, or whether Standby Mode is effectively a power-down condition.

The programmable options are Yes or No.


If set to Yes, then STANDBY will be displayed on the screen and joystick deflections can be used to select different operating Modes. Each of the four joystick directions can be programmed to enter a different Mode. This is set by Standby Forward, Standby Reverse, Standby Left and Standby Right.

If set to No, then STANDBY ON will be displayed on the screen and joystick deflections have no effect. To resume normal operation the Profile Button or an External Profile Switch, if fitted, should be operated.

## 7.24 Standby in Modes

This allows the Standby function to be disabled in particular operating Modes. A typical example is if the R-net is being used to control a PC mouse. In this instance Standby would be undesirable as the joystick is likely to be frequently left in the center position for a period longer than that set by Standby Time.

The programmable options are in the form of a drop-down menu, as shown in the following diagram.

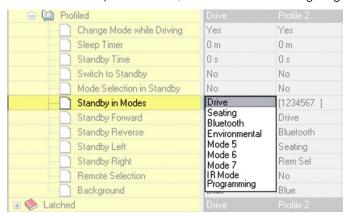


To set whether Standby is enabled in a particular Mode or Modes, simply highlight the appropriate menu item. Any combination of Modes can be selected.

When the drop down menu is closed the selected Modes are expressed as numbers and the parameter appears on the screen as below. The numbers 1 to 8 refer to the Modes listed in the menu.



Example, if Standby is required in Drive and Seating Modes, the following selections should apply.

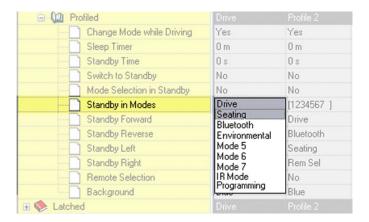

When the drop down menu is closed, it would appear as below.





If a Mode is disabled using Standby in Modes and Standby Time is used to reach a Standby condition, the user must be able to operate either the JSM Mode button or a switch connected to the External Profile Switch Jack to return to Standby.

## 7.25 Standby Forward

This sets which Mode is entered when the joystick is deflected forwards while the system is in Standby Mode. Note, Mode Select in Standby must be set to Yes for this function to be active.

The programmable options are a list of Modes in a drop-down menu. Only one Mode can be selected from the menu.

#### 7.26 Standby Reverse

This sets which Mode is entered when the joystick is deflected in reverse while the system is in Standby Mode. Note, Mode Select in Standby must be set to Yes for this function to be active.

The programmable options are a list of Modes in a drop-down menu. Only one Mode can be selected from the menu.

#### 7.27 Standby Left

This sets which Mode is entered when the joystick is deflected to the left while the system is in Standby Mode. Note, Mode Select in Standby must be set to Yes for this function to be active.

The programmable options are a list of Modes in a drop-down menu. Only one Mode can be selected from the menu.

#### 7.28 Standby Right

This sets which Mode is entered when the joystick is deflected to the right while the system is in Standby Mode. Note, Mode Select in Standby must be set to Yes for this function to be active.

The programmable options are a list of Modes in a drop-down menu. Only one Mode can be selected from the menu.

#### 7.29 Remote Selection - (Remsl)

This sets whether Profiles and Modes can be selected and entered with joystick deflections.

The programmable options are Yes or No.

PROFILE SELECT

2

DRIVE

If set to Yes, then Remote Selection Mode can be entered from the Profile. Entry to Remote Selection Mode is via Standby Mode, the Profile Button or an External Profile Switch. The Profile will also be selectable from Remote Selection Mode.

If set to No, then Remote Selection Mode cannot be entered from the Profile, nor can the Profile be selected from Remote Selection Mode.

In Remote Selection Mode, any other Profile that has Remote Select set to Yes can be selected and entered. The screen display will be as above.


Profiles can be selected with left and right joystick deflections. When the desired Profile or Mode is selected, it can be entered with a forward deflection of the joystick.

If the standby timer expires while the system is in Remote Selection Mode, the Standby Mode will be re-entered and the original Profile will still apply.

## 7.30 Background

This sets the LCD screen background color for each Profile.

The programmable options are Blue and White. Blue is the conventional setting, while White sets a high-contrast screen that may be more easily viewable in very bright conditions. The diagram below shows examples of the two screen types.

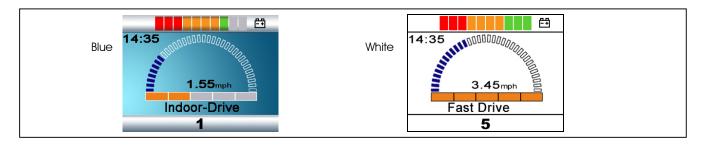

Inputs Devices that have color LCD screens have their own local programming for background, which can be set to Blue, White and Auto. If set to Blue or White, then those settings will always occur, but if set to Auto then the setting as determined by this parameter will occur.

### 8 Latched

# 8.I Latched Drive - (Drive)

This sets the type of latched drive of the wheelchair.

The programmable options are in the form of a drop-down menu as shown in the following diagram.



If set to Off, there will be no latched drive.

If set to Step, the latched drive will be step type, but in the forwards direction only.

If set to Step reverse, the latched drive will be step type in both forwards and reverse.

Stepped drive is when short deflections of the joystick step up or down the latched drive speed of the wheelchair.

If set to Cruise, the latched drive will be cruise type, but in the forwards direction only.

If set to Cruise reverse, the latched drive will be cruise type in both forwards and reverse.

Cruise drive is when the wheelchair accelerates while the joystick is deflected, then, when the joystick is released the drive continues at the achieved speed.



If any Profile has latched drive set, then an Emergency Stop Switch must be fitted to the system via the Profile jack socket on the Joystick Module. If it is required for the system to operate without an Emergency Stop switch, this can be achieved via the parameter, Emergency Stop Switch.



PGDT accepts no liability for losses of any kind resulting from the use of a wheelchair with latched drive or actuator operation and with no normally closed Emergency Stop Switch fitted.

## 8.2 Latched Actuators - (Acts)

This sets whether the seating functions are latched in operation.

The programmable options are Yes and No.

If set to Yes, then seating control will be latched in operation. i.e. a short joystick deflection will start seat movement.

If set to No, then the seating control will not be latched in operation.



If any Profile has latched actuators set, then an Emergency Stop Switch must be fitted to the system via the Profile jack socket on the Joystick Module. If it is required for the system to operate without an Emergency Stop switch, this can be achieved via the parameter, Emergency Stop Switch.



PGDT accepts no liability for losses of any kind resulting from the use of a wheelchair with latched drive or actuator operation and with no normally closed Emergency Stop Switch fitted.

#### 8.3 Latched Timeout - (Time)

This sets the time-out period for latched drive and actuator control.

The programmable range is 0 to 250 Seconds in steps of 1 Second.

The time-out period is the maximum length of time the joystick can be in the neutral position before latched movement ceases. If the period is exceeded, the latched movement will stop. Time-out is therefore an important safety feature that will stop drive or actuator movement should the user be unable to stop the functions in the usual way. It is important that an appropriate time-out period is set.

If a value of 2 or less is set, then the time-out function is disabled. This value should only be used in exceptional circumstances and after all the risks have been fully assessed.



PGDT accepts no liability for losses of any kind resulting from the use of a wheelchair with latched drive or actuator operation and with a Latched Timeout value of less then 2.

## 8.4 Latched Timeout Beep - (Bleep)

This sets whether a beep occurs as the end of the time-out period approaches. The beep is a reminder to the user to give a joystick command to maintain the latched drive or actuator movement.

The programmable options are Yes and No.

If set to Yes, there will be a beep as the timeout period approaches.

If set to No, there will be no beep.

# 9 Seating

#### 9.0.1 Seating Module Enabled

This parameter sets whether a Power Module is compatible with an ISM or with a Seating Module.

The parameter options are Yes or No.

If set to Yes, the Power Module will be compatible with a Seating Module. This setting should also be used if it is intended to use an EL PM90 to drive a single actuator.

If set to No, the Power Module will be compatible with an ISM.

## :: Actuator Axes Explained ::

An actuator axis means a particular function of seat control. An axis may involve just one actuator motor, for example, a simple seat lift; or an axis may involve multiple motors, for example, adjustment of both footrests simultaneously or an anti-shear recline system. There are 6 actuator motor channels on the ISM and an axis can combine any number of these channels.


The R-net system can support up to 12 actuator axes. Each is programmable to define which actuator motors move and also how the user display appears. The relevant parameters are Axis Channels and Axis Display.

## Intelligent Seating Module - Global

## 9.1 Actuator Mode Entry Axis - (Act. Entry Axis)

This sets the actuator axis that is selected when Seating Mode is entered.

The programmable options are in the form of a drop-down menu, as shown in the following diagram.

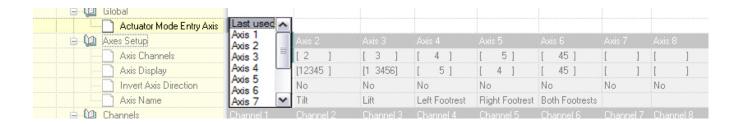

Simply select the required axis. If Last used is selected, this means the axis that was selected on the last occasion Seating Mode was used will be the selected axis when Seating Mode is re-entered.

## Intelligent Seating Module - Axes Setup

#### 9.2 Axis Channels


#### **Axis Display**

#### **Invert Axis Direction**

## These set:

- which actuator motor channels on the ISM will operate for a particular axis.
- the user display for a particular axis.
- whether a channel direction is inverted for a particular axis.

These options are in the form of a dialogue box, as shown in the following example diagram.

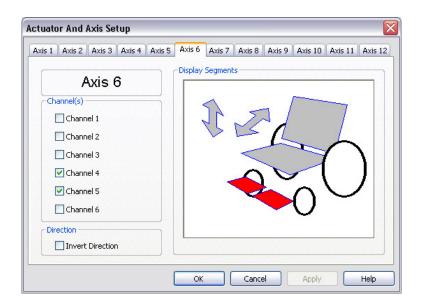


In this example, Axis 6 is the axis that is being set-up. With the programming as shown, Channels 4 and 5 will operate in this axis and the user display on the Joystick Module will highlight those areas colored in red.



The outline of the whole wheelchair symbol is always present on the user display, but with the elements highlighted as defined in red. However, if the arrow symbols are not defined in red, they will not appear on the display.

In this example, the axis direction is not inverted, i.e. when the joystick is deflected forwards, the channel outputs will have pin 2 positive with respect to pin 1.

When the dialogue box is closed, the programming is expressed as follows:

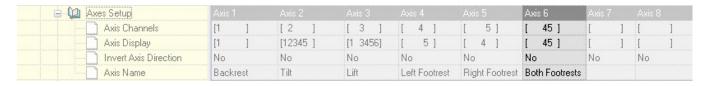

#### 9.3 Axis Name

This parameter sets what text will be displayed on the LCD when the associated Actuator Axis is selected.



This is only applicable on Color Joystick Modules and the Omni.

## Intelligent Seating Module - Channels

## 9.4 Acceleration - (Acc)

This sets the acceleration rate for a particular ISM channel.

Programmable range is 1 to 100 in steps of 1.



This parameter must be set to IOO% on systems that use automatic endstops, i.e. those that do not use micro-switches at the end of actuator travel. If the value is not set to IOO%, then the automatic endstop function will not operate and damage to the actuator motor or the wheelchair frame may occur.

# 9.5 Deceleration - (Dec)

This sets the deceleration rate for a particular ISM channel.

Programmable range is 1 to 100 in steps of 1.



This parameter must be set to IOO% on systems that use automatic endstops, i.e. those that do not use micro-switches at the end of actuator travel. If the value is not set to IOO%, then the automatic endstop function will not operate and damage to the actuator motor or the wheelchair frame may occur.

## 9.6 Up Speed - (UpSpd)

This sets the speed of a particular ISM channel in the Up direction.

Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward and Invert Axis Direction is not set. Programmable range is 25% to 100% in steps of 1%.

## 9.7 Down Speed - (DnSpd)

This sets the speed of a particular ISM channel in the Down direction.

Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced in reverse and Invert Axis Direction is not set. Programmable range is 25% to 100% in steps of 1%.

## 9.8 Up Current Limit

This sets the maximum current output for a particular ISM channel in the Up direction.

Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward and Invert Axis Direction is not set.

The maximum current the ISM can deliver is 15A. However, this can only be achieved if the channel's Up Speed is less than 100%. At a setting of 100% speed, the maximum current is 12A. The actual speed setting that is required is dependent on the electrical characteristics of the actuator motor being driven. Therefore, if currents of greater than 12A are required, the speed settings should be gradually reduced until reliable actuator movement is attained.

Programmable range is 1A to 15A in steps of 0.1A.

This parameter can be important if automatic endstop detection is employed. Automatic endstopping relies on the actuator mechanism being held against a firm mechanical stop at the end of travel. However, some mechanisms may distort over time due to the high endstop force. This force can be effectively reduced by reducing the value of this parameter.



The value must not be set so low that it cannot move the maximum rated load for the actuator mechanism, or does not account for wear and increased friction in the mechanism.

#### 9.9 Down Current Limit

This sets the maximum current output for a particular ISM channel in the Down direction.

Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced in reverse and Invert Axis Direction is not set.

The maximum current the ISM can deliver is 15A. However, this can only be achieved if the channel's Down Speed is less than 100%. At a setting of 100% speed, the maximum current is 12A. The actual speed setting that is required is dependent on the electrical characteristics of the actuator motor being driven. Therefore, if currents of greater than 12A are required, the speed settings should be gradually reduced until reliable actuator movement is attained.

Programmable range is 1A to 15A in steps of 0.1A.

This parameter can be important if automatic endstop detection is employed. Automatic endstopping relies on the actuator mechanism being held against a firm mechanical stop at the end of travel. However, some mechanisms may distort over time due to the high endstop force. This force can be effectively reduced by reducing the value of this parameter.



The value must not be set so low that it cannot move the maximum rated load for the actuator mechanism, or does not account for wear and increased friction in the mechanism.

## 9.10 Endstop Autodetect

This enables or disables the automatic endstop function for a particular actuator channel. Automatic endstop detection is a technique that uses actuator motor current to detect if an endstop has been reached. Upon detection, drive to the actuator motor is automatically cut. This technique removes the need for micro-switches at end limits.

The programmable options are Yes and No.



Automatic endstop can only be used with actuator Acceleration and Deceleration values of IOO%.

SK7798I/5 9I

#### Automatic endstop may not work well if the actuator takes a very low current.

## 9.II Actuator Endstop Timeout

This sets the amount of time the actuator motor current has to be above the endstop detection level (or actuator current limit if Endstop Autodetect is set to No) before the drive is cut. It is actually very rare to have to adjust this parameter, but it can be useful to prevent nuisance stopping, i.e. false detections of endstop.

The programmable range is 0 milliseconds to 500 milliseconds in steps of 10 milliseconds.

# 10 Battery

## IO.I Low Battery Alarm

This sets whether there will be an audible alarm to signal a low battery condition. The point at which the alarm will sound corresponds to the Low Battery Flash Level setting.

The programmable options are Yes and No.

### 10.2 Low Battery Flash Level

This sets the point at which the battery gauge on the LCD screen starts to flash slowly to warn of a low battery condition.

The programmable range is 1 to 10 in steps of 1, and corresponds to the number of bars shown on the battery gauge. For example, if this value is set to 2, then the flashing will occur when the gauge drops to 2 bars.

## IO.3 Cable Resistance

This calibrates the system to compensate for the electrical resistance of the battery cable and connectors to the Power Module. This is necessary to ensure an accurate estimation of battery charge.

The programmable range is 0mOhms to 500mOhms in steps of 1mOhm.

The value should take into account the electrical resistance in both the positive and negative paths, including connectors.

## IO.4 Calibration Factor

This allows further fine calibration of the battery gauge. This is normally set at the factory and should not need further adjustment.

### IO.5 Low Voltage Cut-out, Low Voltage Time

These two parameters allow a low battery voltage cut-out function to be programmed. Such a function can be used to prevent over-discharge of the batteries.

If the battery voltage is below the value set by Low Voltage Cut-out for a period of time longer than that set by Low Voltage Time, then drive will be prevented. This will be recorded in the Power Module's error log.

Drive will then be permitted to resume when the battery voltage rises above the value of Low Voltage Cut-out.

The programmable range for Low Voltage Cut-out is 16V to 22V in steps of 0.5V.

The programmable range for Low Voltage Time is 1s to 255s in steps of 1s.

## II General

## II.I Brake Disconnected Alarm

This sets whether there is an audible alarm when one or both of the brakes are disconnected from the Power Module. The programmable options are Yes and No.

## II.2 Brake Fault Detect

This sets whether the Power Module will detect electrical faults in the wheelchair's brakes or the connection to them.

The programmable options are Yes and No.



This parameter should only ever be set to No if there are no electrical brakes fitted to the wheelchair.

### II.3 Brake Voltage

This sets the voltage output from the Power Module to the wheelchair's electrical brakes.

The programmable options are 12V and 24V.



It is essential the correct value for the brakes' operating voltage is set. An incorrect value could result in damage to the brakes or motors. PGDT accepts no liability for losses of any kind resulting from incorrect programming of this parameter.

## II.4 Soft Stop Rate

This sets the deceleration rate that is used while the wheelchair is "soft-stopping". "Soft-stopping" is if the system is powered-down while the wheelchair is being driven.

The programmable range is 0 to 200 in steps of 1.

If this value is the same or lower than the Forward Deceleration value, then the latter value will be used for "soft-stopping".



It is the wheelchair manufacturer's responsibility to ensure that the wheelchair complies with the correct legislation for braking distances for the country in which the wheelchair is to be sold. PGDT accepts no liability for losses of any kind resulting from incorrect programming of this parameter.

## II.5 Front Wheel Drive

This selects specialist software for front wheel driven wheelchairs.

The programmable range is 0% to 100% in steps of 1%.

If set to 0%, then "normal" software for rear wheel driven wheelchairs is employed.

If set to any other value then front wheel drive software is employed. A higher value makes the software more suitable for faster wheelchairs.

## II.6 Seat Reversal

This can be used to set the R-net to function with wheelchairs that have a seat reversal function, i.e. the seat can be rotated to exchange the wheelchair configuration between front wheel drive and rear wheel drive. It is typical on such wheelchairs to have

a micro-switch that signals which configuration the wheelchair is actually in. This micro-switch should be connected to the Inhibit 2 input on the Power Module.

There are three programmable options – Never, Inhibit 2 at 0V and Inh 2 at 0V - Profile mask active.

If set to Never, then the Inhibit 2 input is not related to the seat reversal function.

If set to Inhibit 2 at 0V and the Inhibit 2 input is connected to 0V, then the wheelchair will be considered to be in a rear wheel drive configuration and the following conditions will apply.

The value of Front Wheel Drive Rate will be forced to 0 within the Power Module's software.

The Invert M1 Direction, the Invert M2 Direction and the Motor Swap parameters will be toggled within the Power Module's software

If set to Inhibit 2 at 0V and the Inhibit 2 input is not connected to 0V, then the wheelchair will be considered to be in a front wheel drive configuration and the following conditions will apply.

The programmed value of Front Wheel Drive Rate will apply.

The programmed settings of Invert M1 Direction, the Invert M2 Direction and the Motor Swap parameters will apply.

If set to Inh 2 at 0V - Profile mask active and the Inhibit 2 input is connected to 0V, then the wheelchair will be considered to be in a rear wheel drive configuration and the following conditions will apply.

The value of Front Wheel Drive Rate will be forced to 0 within the Power Module's software.

The Invert M1 Direction, the Invert M2 Direction and the Motor Swap parameters will be toggled within the Power Module's software.

Only the Profiles that have Seat Reversal Profile set to Yes will be available.

If set to Inhibit 2 at 0V and the Inhibit 2 input is not connected to 0V, then the wheelchair will be considered to be in a front wheel drive configuration and the following conditions will apply.

The programmed value of Front Wheel Drive Rate will apply.

The programmed settings of Invert M1 Direction, the Invert M2 Direction and the Motor Swap parameters will apply.

Only the Profiles that have Seat Reversal Profile set to No will be available.

## II.7 Charger Inhibit 3 Enable

This should be set if it is intended to use the Inhibit 3 input on the Power Module as an On-Board Charger (OBC) connection.

The programmable options are Off, High to Drive and Low to Drive.

If set to Off, then inhibit 3 is not used as a Charger inhibit.

If set to High to Drive, Drive is inhibited when in Band 0.

If set to Low to Drive, Drive is inhibited when in Band 3.



Regardless of the setting of this parameter, the Inhibit 3 input can still be used as a multi-state input to provide speed limits and/or actuator inhibits. See section I3, Inhibits, for more details.

## II.8 Charger Inhibit 3 Latching

This sets whether the On-Board Charger inhibit, which is located in the 3-way charger socket on the Power Module is latching or non-latching.

Latching means that power must be cycled after the inhibit is removed, in order to resume normal operation.

Non-latching means that normal operation will resume immediately the inhibit is removed.

The programmable options are Yes and No.

If set to Yes, the operation is latching.

If set to No, the operation is non-latching.



Regardless of the setting of this parameter, the Inhibit 3 input can still be used as a multi-state input to provide speed limits and/or actuator inhibits. See section I3, Inhibits, for more details.

## II.9 Charger Inhibit I Latching

This sets whether the Off-Board Charger inhibit, which is located in the 3-way charger socket on the Joystick Module is latching or non-latching.

Latching means that power must be cycled after the inhibit is removed, in order to resume normal operation.

Non-latching means that normal operation will resume immediately the inhibit is removed.

The programmable options are Yes and No.

If set to Yes, the operation is latching.

If set to No, the operation is non-latching.

## II.IO Charger Inhibit I Alarm

This sets whether an alarm sounds if there is a charge inhibit present on Inhibit 1.

A charge inhibit is invoked if an Off-Board Charger is connected into the 3-way charger socket on the Joystick Module.

The programmable options are On and Off.

#### II.II Charger Inhibit 3 Alarm

This sets whether an alarm sounds if there is a charge inhibit present on Inhibit 3.

A charge inhibit is invoked if an On-Board Charger is connected to the OBC connector on the Power Module and Charger Inhibit 3 Enable is set to Yes.

The programmable options are On and Off.

## 12 Motor

## Motor - Global

#### I2.I Maximum Current Limit. Boost Drive Current. Boost Drive Time

These three parameters set the maximum current output from the Power Module. They are closely related, so are detailed in one section.

Maximum Current Limit sets the long-term maximum current output of the Power Module.

The programmable range is from 50A to 60A on a PM60, 50A to 80A on a PM80 or ELPM90 and 50A to 100A on a PM120, in steps of 1A.


Boost Drive Current sets the short-term maximum current output of the Power Module.

The programmable range is from the setting of Maximum Current Limit to: 60A on a PM60, 80A on a PM80, 90A on an ELPM90 and 120A on a PM120, in steps of 1A.

Boost Drive Time sets the period of time that the level of current set by Boost Drive Current is available.

The programmable range is from 0 Seconds to 10 Seconds in steps of 1 Second.

The following graph shows the relationship between the parameters.

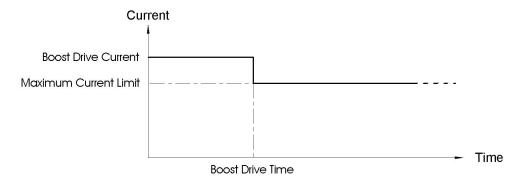

There are maximum permissible values, which must not be exceeded, for each of these parameters. These values are shown in the table below.

	PM60	PM80	PM90	PM120
Maximum Current Limit	60 Amps	80 Amps	80 Amps	100 Amps
Boost Drive Current	60 Amps	80 Amps	90 Amps	120 Amps
Boost Drive Time			10 Secs	10 Secs



The values in the table above should never be exceeded. Doing so will invalidate the warranty and affect the long-term reliability of the Power Module.

## 12.2 Current Foldback Threshold, Current Foldback Time, Current Foldback Level

These three parameters can be used to protect the wheelchair motors from overheating. The parameters are closely related, so are dealt with in one section.

Current Foldback Time sets a period of time, which if exceeded by the software timer will initiate a reduction in the Power Module's maximum current output.

The programmable range is 0 Seconds to 250 Seconds in steps of 1 Second.

Current Foldback Level sets the level of current output reduction, as a percentage of Maximum Current Limit, that will result.


The programmable range is 25% to 100% in steps of 1%.

Current Foldback Threshold sets a level of current, which, if exceeded, will start a software timer running. If the current drops below the level set by Current Foldback Threshold, then the software timer will decrement each second.

The programmable range is 10A to Maximum Current Limit (see previous section) in steps of 1A.

After the reduction has occurred, the maximum current output will remain at the reduced level for a period of 5 times Current Foldback Time. This is a typical period that is sufficient to allow motors to cool.

The graph following shows the relationship between the parameters.

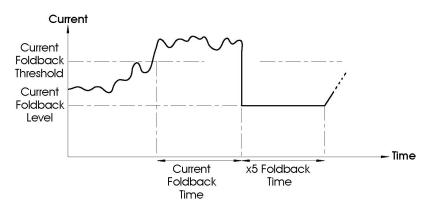

Example: Maximum Current Limit is set to 80A

Current Foldback Threshold is set to 50A

Current Foldback Time is set to 20 Seconds

Current Foldback Level is set to 50%

If the motor current is above 50A for a period of 20 Seconds, then it will reduce to 50% of 80A = 40A. It will then remain at this reduced level for  $5 \times 20$  Seconds = 100 Seconds.

## 12.3 Compensation

This matches the Power Module to the motors used on the wheelchair. This is necessary to achieve optimal performance and control, especially on a gradient.

The programmable range is 0mOhms to 500mOhms in steps of 5mOhms.

PGDT recommend this value is initially set to 70% of the total resistance of the motor's armature, and all the connectors and cables between the motor and the Power Module motor output. Motor manufacturers should be able to supply a figure for the motor's armature resistance and typical connector and cables may be around 40mOhm.

Example:

The motor has a specified armature resistance of 100mOhms, and cables and connectors between the motor and the Power Module are estimated to be 40mOhm. This gives a total value of 140mOhm. The value of Compensation should therefore be set to 70% of 140mOhm = 100mOhm.

Once this value has been programmed, some drive testing should be carried out.

First of all, check that the wheelchair is controllable and comfortable to drive on a smooth, level surface. If it is not, then the value of Compensation may be too high and should be reduced.

Then drive the wheelchair towards a gradient at a very low speed, perhaps 10%. There should be no reduction in speed as the wheelchair climbs the gradient. If the speed drops or the wheelchair stops, then the value of Compensation may be too low and should be increased.

As a final check, drive the wheelchair up a gradient at full speed and release the joystick. The wheelchair should come to a smooth, complete stop and should not rollback before the brakes are applied. If the wheelchair does roll-back, then try

increasing the value of Compensation slightly. If the wheelchair appears to drive slightly up the gradient before the brakes are applied, then the value of Compensation should be reduced slightly.

All these tests should be conducted when the motors are cold and hot.



These tests should be conducted in an open space and a restraining device such as a set belt should always be used. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The wheelchair manufacturer is responsible for ensuring that the value of Compensation is correctly set for the motors being used on the wheelchair. This is particularly important if replacement motors are being sent out to upgrade a wheelchair already in the field. Failure to comply with this requirement may result in poor control characteristics, which in extreme instances can make a wheelchair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The wheelchair manufacturer is responsible for always ensuring that any replacement motors supplied as spare parts are fully compatible with the originals that the system was programmed to match. Failure to comply with this requirement may result in poor control characteristics, which in extreme instances can make a wheelchair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



A control system must not be moved from one wheelchair type to another without first ensuring that the system is correctly programmed for the wheelchair it is intended to fit it to. Even if control systems have the same PGDT part number, they may not be programmed the same. Failure to ensure the system is correctly programmed for the wheelchair may result in a hazardous condition. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

## I2.4 Invert MI Direction

This inverts the direction of rotation of motor channel M1 on the Power Module. If the parameter, Motor Swap, is set to No, then channel M1 drives the left motor.

The programmable options are Yes or No.

If set to Yes, then when the joystick is deflected forwards, M1+ pin will be positive with respect to M1- pin.

If set to No, then when the joystick is deflected forwards, M1+ pin will be negative with respect to M1- pin.

## 12.5 Invert M2 Direction

This inverts the direction of rotation of motor channel M2 on the Power Module. If the parameter, Motor Swap, is set to No, then channel M2 drives the right motor.

The programmable options are Yes or No.

If set to Yes, then when the joystick is deflected forwards, M2+ pin will be positive with respect to M2- pin.

If set to No, then when the joystick is deflected forwards, M2+ pin will be negative with respect to M2- pin.

## I2.6 Motor Swap

This swaps the motor output channels, M1 and M2, of the Power Module.

The programmable options are Yes or No.

If set to Yes, then channel M1 drives the right motor and channel M2 drives the left motor.

If set to No, then channel M1 drives the left motor and channel M2 drives the right motor.

#### 12.7 Steer Correct

This parameter compensates for any mis-match in motor speeds, thereby ensuring the wheelchair drives in a straight line when the joystick is being pushed directly forward. This is particularly useful for switch type Input Devices.

The programmable range is -9 to 9 in steps of 1.

If the wheelchair is veering to the left, then the value should be increased. Conversely, if the wheelchair is veering to the right, then the value should be decreased.



This logic may be affected by the setting of parameter, Motor Swap, which, if set to Yes, will reverse the logic.

#### 12.8 Display Speed - (Display Speed)

This sets whether the speedometer is displayed in miles per hour or kilometers per hour. Note, the Power Module always calculates and transmits speed information in miles per hour.

The programmable options are Off, mph or km/h.

If set to Off, there will be no speed or odometer is displayed on the LCD screen.

If set to mph, then the speed and distance will be displayed as the value transmitted by the Power Module (which always sends the information in miles per hour) and the text 'mph' will appear on the LCD screen.

If set to km/h, then the module with the speed and odometer display will convert the Power Module value into kilometers per hour and the text 'km/h' will appear on the LCD screen.



Only certain Input Devices, e.g. CJSM or Omni, have the speedometer and Odometer functions.

## 12.9 Maximum Rated Speed

This calibrates the speedometer and odometer functions. The value chosen should correspond to the speed of the wheelchair that is expected at a setting of Maximum Forward Speed = 100%. This value is always set in miles per hour.

The programmable range is 0.0mph to 20.0mph.



Only certain Input Devices, e.g. CJSM or Omni, have the speedometer and odometer functions. If the parameter is set to zero the Speed and Odometer display will not be visible

## 12.10 Maximum Displayed Speed (Max Display Speed)

This sets the operation of the graphical speed display, i.e. the blue semi-circle on the CJSM and Omni.

Ordinarily, the powerchair will be dispatched so that the graphical speed display reaches its maximum point at the maximum rated speed of the powerchair. This means that, if the powerchair is later programmed to a lower speed, to suit a particular user for example, the graphical speed display would never reach its maximum. However, this can be compensated for by setting the value of Maximum Displayed Speed. This is best explained in an example.

A powerchair with a rated speed of 4mph requires the Maximum Forward Speed to be re-programmed from 100% to 80%, in order to suit a particular user. This would mean the powerchair now has a top speed of 3.2mph and that the graphical speed display will never 'fill' beyond a point of 80%. If it is required for the graphical speed display to 'fill' to 100%, then the value of Maximum Displayed Speed should be set to 3.2mph

The programmable range is 0.0mph to 20.0mph.



Only certain Input Devices, e.g. CJSM or Omni, have the speedometer and Odometer functions.

#### Motor - Profiled

## 12.11 Compensation Factor

This allows different values of Compensation to be used across different profiles.

The programmable range is 0% to 200% in steps of 1%.

The programmed value adjusts the actual value of the parameter, Compensation.

For example, if Compensation was set to 100mOhms and Compensation Factor for a particular Profile was set to 150%, then an effective value of 150mOhms would be used for that Profile. Likewise, if in another Profile, Compensation Factor was set to 80%, then an effective value of 80mOhms would result.

See section, Compensation for details of how to set that parameter. Compensation must be set accurately before making adjustments to Compensation Factor.



Inappropriate values of Compensation and Compensation Factor could make the wheelchair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind resulting from inappropriate setting of either or both of these parameters.

## 13 Inhibits

## :: Inhibits Explained ::

Both the Power Module and the Intelligent Seating/light Module (ISM) each have two Inhibit Inputs. On the Power Module, these are referred to as Inhibit 2 and Inhibit 3; and on the ISM, these are referred to as Inhibit 4 and Inhibit 5. For connection details of these inputs, please refer to the Installation Chapter.

Each of these inputs can recognize four different input conditions (or states). This means that four different events can originate from a single input. See the following Speed Inhibit and Actuator Inhibit Examples.

The state of an Inhibit input is defined as a percentage and is set by the resistance connected to the input.

If the input is connected directly to OV, then it is considered to be at 0%.

If the input is connected to 0V via a 10KOhm resistor or greater, then it is considered to be at 100%.


Each of the 99 steps between corresponds to 1000hm. As an example, if the input is connected to 0V via a 4.7KOhm resistor, then the input is considered to be at 47%.

Dependent on the state of an input, it is in one of four bands. The size of these bands is programmable via the parameters Lower Level Threshold, Middle Level Threshold and Upper Level Threshold. These are shown in the following diagram.

Each of the Threshold Level parameters can be set between 0% and 100% in steps of 1%.

After the input bands have been defined, then each input can be linked to a speed limit or full drive inhibit via the parameters: Assign, Speed Limit in Band 0, Speed Limit in Band 1, Speed Limit in Band 2 and Speed Limit in Band 3.

Assign, Speed Limit in Bands 0, 1, 2 and 3 as Operation.

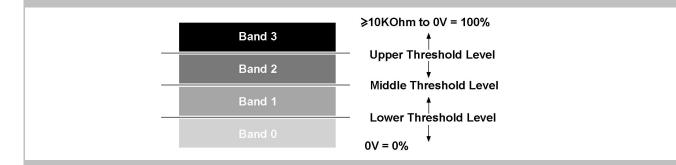


Each input can also be linked to an actuator function via the parameters: Assign, Channel x Up: Inhibit in Bands and Channel x Down: Inhibit in Bands.

## :: Speed Inhibit Example ::

**Application:** A wheelchair with an elevating seat, whereby, when the seat is lowered, full speed drive is permitted. Then as the seat raises, 3 switches are operated, thereby changing the resistance connected to the input and giving, say, 50% speed limit to 25% speed limit to full drive inhibit.

In this example the Inhibit is connected to Inhibit 2 input that is sited on the Power Module.

A circuit such as follows is used.

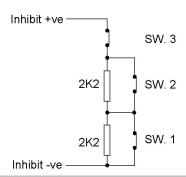


The Inhibit input must be assigned to a specific software Inhibit. As this is a speed inhibit, the parameter Drive Inhibits Assign should be used. Inhibit 2 input is therefore assigned to Drive Inhibit A.

When the seat is in the fully lowered position, all switches are closed and the electrical resistance is a short-circuit or 0 Ohms. As the seat raises, SW1 opens giving 2.2KOhms, then SW2 opens giving 4.4KOhms and at full height SW3 opens giving a complete open-circuit.

Programming would be as follows:

Inputs PM - Threshold Levels:	
Upper Level Threshold	54%
Middle Level Threshold	34%
Lower Level Threshold	12%

This gives a band arrangement as follows.

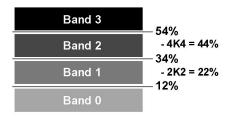

As good practice, the threshold limits have been chosen so the actual resistance values that trigger a change are in the approximate center of the bands, thereby eliminating the risk of a system erroneously switching between bands.

The 4 Speed Limits for Drive Inhibit A would need to be set as below:

#### Inhibit A:


Speed Limit in Band 0	100%	
Speed Limit in Band 1	50%	
Speed Limit in Band 2	25%	
Speed Limit in Band 3	0%	

IO2 SK7798I/6

#### :: Actuator Inhibit Examples ::

**Application 1**: A wheelchair is fitted with a reclining backrest that has endstop switches connected to the R-net's Inhibit inputs 4 and 5 on the ISM. In this example the switches are installed to be normally closed and the Backrest has been programmed to use Actuator Channel 2.

A circuit such as follows is used.

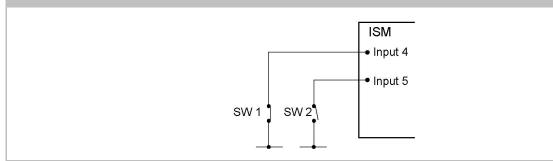


The Inhibit input must be assigned to a specific software Inhibit. As this is an Actuator Inhibit, the parameters Actuator Inhibits Channel X - Up: Assign & Down: Assign should be used.

The inputs are therefore assigned to Actuator Inhibits Channel 2 as follows:

Up: Assign - Input 4 Down: Assign - Input 5

As these inputs are connected to the ISM the Threshold levels must be set in the parameter Inputs - ISM.

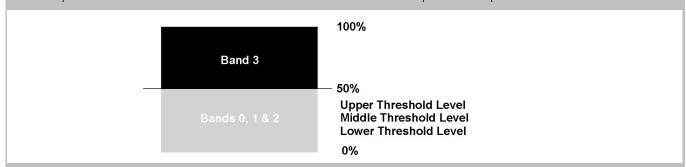

When Actuator Inhibits Channel 2 is programmed in the following way the inhibit will effectively stop the backrest actuator being moved in the Up direction when Switch 1 is open and in the Down direction when Switch 2 is open.

Up: Assign 4		Down: Assign 5	
Inhibit in Band 0	no	Inhibit in Band 0	no
Inhibit in Band 1	no	Inhibit in Band 1	no
Inhibit in Band 2	no	Inhibit in Band 2	no
Inhibit in Band 3	yes	Inhibit in Band 3	yes

**Application 2:** A wheelchair is fitted with a reclining seat. It is required to offer a programmable angle of recline to suit different users. A potentiometer of maximum value 10KOhm is fitted to the recline mechanism and is connected to the R-net's Inhibit 4 input sited on the ISM. In this example the Backrest has been programmed to use Actuator Channel 3.


The Inhibit input must be assigned to a specific software Inhibit. As this is an Actuator Inhibit, the parameters Actuator Inhibits Channel 3 - Up: Assign & Down: Assign should be used.

The inputs are therefore assigned to Actuator Inhibits Channel 3 as follows:

SK7798I/5 IO3

Up: Assign - Input 4 Down: Assign - Input 4

As these inputs are connected to the ISM the Threshold levels must be set in the parameter Inputs - ISM.

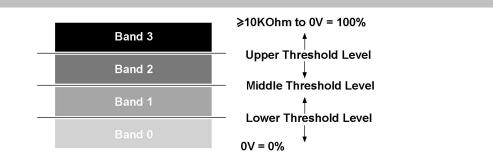

By programming the Actuator Channel 3 Up Inhibit and Actuator Channel 3 Down Inhibit parameters as below, it is possible to program the maximum angle of travel of the recline mechanism via the Inhibit 4.

#### **Actuator Channel 3**

Up: Assign 4		Down: Assign 4	
Inhibit in Band 0	yes	Inhibit in Band 0	no
Inhibit in Band 1	no	Inhibit in Band 1	no
Inhibit in Band 2	no	Inhibit in Band 2	no
Inhibit in Band 3	no	Inhibit in Band 3	yes

This programming means that the Up movement will be inhibited when the Inhibit 4 input is in Band 0, i.e. one end of recline travel, and that the Down movement will be inhibited when the Inhibit 4 input is in Band 3, i.e. the other end of the recline travel. Referring the Band diagram above, it can be seen that by programming Inhibit 4 Upper Level Threshold and Inhibit 4 Lower Level Threshold, then the actual angular range of the recline mechanism is adjusted.

## Inhibits - Inputs - PM

## I3.I Lower Level Threshold

This sets the level of the threshold between Band 0 and Band 1 for a particular Inhibit Input (Inhibit 2 or Inhibit 3) on the Power Module. See :: Inhibits Explained :: for more details on Power Module inhibits and Bands.

The programmable range is 0% to 100% in steps of 1%.

#### 13.2 Middle Level Threshold

This sets the level of the threshold between Band 1 and band 2 for a particular Inhibit Input (Inhibit 2 or Inhibit 3) on the Power Module. See :: Inhibits Explained :: for more details on Power Module inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

#### 13.3 Upper Level Threshold

This sets the level of the threshold between Band 2 and band 3 for a particular Inhibit Input (Inhibit 2 or Inhibit 3) on the Power Module. See :: Inhibits Explained :: for more details on Power Module inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

IO4 SK7798I/6

#### **I3.4** Debounce Time

This sets a debounce time for the Inhibit input. Debounce is the amount of time the input must be stable before a new condition is registered. This is particularly useful if mercury switches are being employed and eliminates the effect of "splashing" across the contacts.

The programmable range is 0mS to 400mS in steps of 50mS.

#### Inhibits - Inputs - ISM

#### **13.5** Lower Level Threshold

This sets the level of the threshold between Band 0 and Band 1 for a particular Inhibit Input (Inhibit 4 or Inhibit 5) on the ISM. See :: Inhibits Explained :: for more details on ISM inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

## 13.6 Middle Level Threshold

This sets the level of the threshold between Band 1 and band 2 for a particular Inhibit Input (Inhibit 4 or Inhibit 5) on the ISM. See :: Inhibits Explained :: for more details on ISM inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

#### 13.7 Upper Level Threshold

This sets the level of the threshold between Band 2 and band 3 for a particular Inhibit Input (Inhibit 4 or Inhibit 5) on the ISM. See :: Inhibits Explained :: for more details on ISM inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

# I3.8 Debounce Time

This sets a debounce time for the Inhibit input. Debounce is the amount of time the input must be stable before a new condition is registered. This is particularly useful if mercury switches are being employed and eliminates the effect of "splashing" across the contacts.

The programmable range is 0mS to 400mS in steps of 50mS.

## Inhibits - Drive Inhibits

## I3.9 Assign

The R-net Power Module has five drive separate software Drive Inhibits. These are referred to as A, B, C, D and E; and each of these can be initiated by any actual hardware inhibit input in the system. This functionality means that speed limits or drive inhibits can be triggered from 5 individual sources.

The parameter, Assign, links the software inhibits to an actual hardware inhibit. For example, if Assign: A is set to 2, then the operation of Drive Inhibit A will be controlled via Inhibit Input 2, which is located on the Power Module. Alternatively, if Assign: A is set to 4, then the operation of Drive Inhibit A will be controlled via Inhibit Input 4, which is located on the Intelligent Seating/Lighting Module.

The programmable options are Off and 2 to 33.

If set to Off, then the relevant Drive Inhibit can never be active.

If set to a numeric value, then the relevant Drive Inhibit will be under the control of the Inhibit Input that corresponds to that number. Inhibit Inputs 2 and 3 are on the Power Module, Inhibits 4 and 5 are on the Intelligent Seating/Lighting Module, and Inhibits 6 to 33 will be on other PGDT or third party modules.

SK7798I/5 IO5

#### I3.IO Speed Limit in Band O

This sets the maximum speed of the wheelchair when the Inhibit input, which is assigned to the Drive Inhibit, is in Band 0. See :: Inhibits Explained :: for more details on Power Module inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

If set to 0%, then a full drive inhibit will be effected and the wheelchair's brakes will not release. See also sections, Operation and Inhibit Alarm. below.

If set to between 1% and 100%, the value will correspond to the maximum possible speed the wheelchair can drive.

## I3.II Speed Limit in Band I

This sets the maximum speed of the wheelchair when the Inhibit input, which is assigned to the Drive Inhibit, is in Band 1. See :: Inhibits Explained :: for more details on Power Module inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

If set to 0%, then a full drive inhibit will be effected and the wheelchair's brakes will not release. See also sections, Operation and Inhibit Alarm, below.

If set to between 1% and 100%, the value will correspond to the maximum possible speed the wheelchair can drive.

#### 13.12 Speed Limit in Band 2

This sets the maximum speed of the wheelchair when the Inhibit input, which is assigned to the Drive Inhibit, is in Band 2. See :: Inhibits Explained :: for more details on Power Module inhibits and bands.

The programmable range is 0% to 100% in steps of 1%.

If set to 0%, then a full drive inhibit will be effected and the wheelchair's brakes will not release. See also sections, Operation and Inhibit Alarm, below.

If set to between 1% and 100%, the value will correspond to the maximum possible speed the wheelchair can drive.

## 13.13 Speed Limit in Band 3

This sets the maximum speed of the wheelchair when the Inhibit input, which is assigned to the Drive Inhibit, is in Band 3. See :: Inhibits Explained :: for more details on Power Module inhibits and bands. The programmable range is 0% to 100% in steps of 1%.

If set to 0%, then a full drive inhibit will be effected and the wheelchair's brakes will not release. See also sections, Operation and Inhibit Alarm, below.

If set to between 1% and 100%, the value will correspond to the maximum possible speed the wheelchair can drive.

# **I3.14** Operation

This sets whether a full drive inhibit, i.e. 0% speed limit setting, is a latching or non-latching function.

The programmable options are Non-latching and Latching.

If set to Non-latching, then as soon as the signal that initiated the drive inhibit is removed, then drive can resume.

If set to Latching, then to resume drive after the signal that initiated the drive inhibit is removed, then the system power must first be cycled.

## 13.15 Inhibit Alarm

This sets whether an audible alarm will sound in a full drive inhibit condition, i.e. speed limit setting of 0% is active.

The programmable options are Yes and No.

If set to Yes, then an alarm will sound while the full drive inhibit is present.

If set to No, then no alarm will sound.

IO6 SK7798I/6

#### Inhibits - Actuator Inhibits

# I3.16 Channel x - Up: Assign (x = 1 to 6)

The R-net Intelligent Seating/Lighting Module has three separate software inhibits for the up direction of each of its output channels, 1 to 6. Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward and Invert Axis Direction is not set. These inhibits are referred to as A, B and C; and each can be initiated by any actual hardware input in the system. This functionality means that any actuator channel direction can be inhibited from 3 individual sources.

The parameter, Assign, links the software inhibits to an actual hardware inhibit. For example, if Assign: A is set to 4, then the operation of Channel x Up Inhibit will be controlled via Inhibit Input 4, which is located on the Intelligent Seating Module.

The programmable options are Off and 2 to 33.

If set to Off, then the relevant Channel x Up Inhibit can never be active.


If set to a numeric value, then the relevant Channel x Up Inhibit will be under the control of the Inhibit Input that corresponds to that number. Inhibit Inputs 2 and 3 are on the Power Module, Inhibits 4 and 5 are on the Intelligent Seating/Lighting Module, and Inhibits 6 to 33 will be on other PGDT or third party modules.

# I3.17 Channel x - Up: Inhibit in Bands (x = I to 6)

This sets the condition of the input (assigned to Channel x Up Inhibit) that will result in an inhibit of the actuator drive in the up direction. Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward and Invert Axis Direction is not set. The inhibit can be set to occur in any combination of bands. See 13 Inhibits Explained for more details on Bands.

The programmable options are Band 0, Band 1, Band 2 and Band 3.

If the parameter is selected, then a drop down menu as below will appear and the bands required to invoke the inhibit should be selected. For example, if the inhibit is required for Band 3 only, then that option should be selected, as shown.

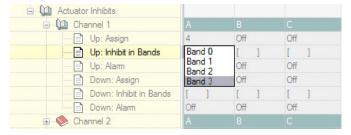

When the drop down menu is closed, it would appear as below.



## I3.18 Channel x - Up: Alarm

This sets whether an audible alarm will sound when a channel's up direction in inhibited. Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward and Invert Axis Direction is not set.

The programmable options are On and Off.

If set to On, then an alarm will sound while an inhibit is present.

If set to Off, then no alarm will sound.

SK7798I/5 IO7

## I3.19 Channel x - Down: Assign(x = I to 6)

The R-net Intelligent Seating/Lighting Module has three separate software inhibits for the down direction of each of its output channels, 1 to 6. Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced in reverse and Invert Axis Direction is not set. These inhibits are referred to as A, B and C; and each can be initiated by any actual hardware input in the system. This functionality means that any actuator channel direction can be inhibited from 3 individual sources.

The parameter, Assign, links the software inhibits to an actual hardware inhibit. For example, if Assign: A is set to 4, then the operation of Channel x Down Inhibit will be controlled via Inhibit Input 4, which is located on the Intelligent Seating Module.

The programmable options are Off and 2 to 33.

The programmable options are Off and 2 to 33.

If set to Off, then the relevant Channel x Down Inhibit can never be active.

If set to a numeric value, then the relevant Channel x Down Inhibit will be under the control of the Inhibit Input that corresponds to that number. Inhibit Inputs 2 and 3 are on the Power Module, Inhibits 4 and 5 are on the Intelligent Seating/Lighting Module, and Inhibits 6 to 33 will be on other PGDT or third party modules.

# I3.20 Channel x - Down: Inhibit in Bands (x = I to 6)

This sets the condition of the input (assigned to Channel x Down Inhibit) that will result in an inhibit of the actuator drive in the down direction. Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced in reverse and Invert Axis Direction is not set. The inhibit can be set to occur in any combination of bands. See 13 Inhibits Explained for more details on bands.

The programmable options are Band 0, Band 1, Band 2 and Band 3.

If the parameter is selected, then a drop down menu as below will appear and the bands required to invoke the inhibit should be selected. For example, if the inhibit is required for Band 3 only, then that option should be selected, as shown.



When the drop down menu is closed, it would appear as follows.



## 13.21 Channel x - Down: Alarm

This sets whether an audible alarm will sound when a channel's down direction in inhibited. Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced in reverse and Invert Axis Direction is not set.

The programmable options are On and Off.

If set to On, then an alarm will sound while an inhibit is present.

I08 SK7798I/6

If set to Off, then no alarm will sound.

## 14 Lights

## 14.1 Lamp Voltage

This sets the operating voltage for the wheelchair's lighting system.

The programmable options are 12V and 24V. The wheelchair's light and indicators should be fitted with lamps of the corresponding value.

If set to 12V, then the lights and indicators will remain a constant brightness, whilst the battery voltage is above this level

If set to 24V, then the brightness of the lamps and indicators will vary with fluctuations in the battery voltage.



TÜV Germany requires the I2V constant brightness condition.

#### 14.2 Indicator Fault Detection

This sets whether defective turn indicator lamps can be detected. The detection circuitry is looking for the presence of 2 x 21W lamps per side.

The programmable options are Yes and No.

If set to Yes, then if  $2 \times 21 \text{W}$  lamps (per side) in working order are not connected to the Seating/lighting Module, a turn indicator fault will be detected and indicated to the user (via a rapid flash of the relevant turn indicator on the Joystick Module).

If set to No, then the indicator fault detection circuitry will be turned off. This option should be used if  $2 \times 21 \text{W}$  lamps (per side) are not being used for the turn indicators.



TÜV Germany requires that 2 x 2IW lamps per side are used for the turn indicators.

#### 14.3 Swap Indicators Seat Reversal

This sets whether the left and right turn indicator channels are exchanged when the seat is reversed. The seat is considered to be reversed when the Inhibit 2 input on the Power Module is at 0V. For more details on seat reversal, please refer to section, Seat Reversal.

The programmable options are Yes and No.

If set to Yes, then if the seat is reversed, the turn indicator left and right channels on the Seating/Lighting Module will be exchanged. This option should be used if the turn indicators are mounted on the wheelchair frame.

If set to No, then if the seat is reversed, the turn indicator left and right channels will be unaffected. This option should be used if the turn indicators are mounted on the wheelchair seat.

#### 14.4 Brake Lights or Horn

This sets whether the brake light output on the Seating/Lighting Module is to be used for Brake lights or a Horn function.

The programmable options are Brake Light and Horn.

If set to Brake Light then the brake light function will be enabled.

If set to Horn, then the Horn function will be enabled.

# IS OEM Factory

The parameters within the OEM Factory group allow the manufacturer to set a maximum and minimum programming area for a set of parameters. The set values create the new programmable range for the vehicles parameters which can not be adjusted without the R-net PC Programmer OEM level.

Example: A standard range for parameter X is 1% to 100%

The Absolute X parameter is set at 15% min to 75% max

This means that anyone wishing to change parameter X from now on has only got an available range of 15 to 75%.



It is the responsibility of the vehicle manufacturer, to ensure that adequate testing of the vehicle is carried out, to ensure that the values set will maintain vehicle stability and user safety. PGDT accepts no liability for losses of any kind resulting from this parameter being incorrectly programmed.

IIO SK7798I/6



CHAPTER 4 - ELPM 90, JSM-LED & SM

SK77981/5 III

II2 SK7798I/6


## I Introduction

This section describes the differences between the previous sections if any of the following modules are used. It is therefore important that all the previous sections have been read and understood.

JSM-LED LED Joystick Module

JSM-LED-L LED Joystick Module with lighting control buttons

EL PM90 EL 90A Power Module SM Seating Module

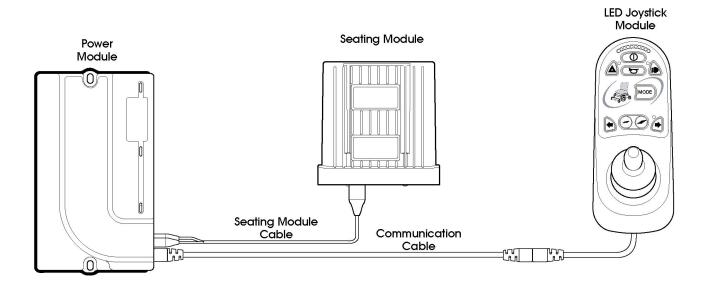



The EL PM90 can actually be used with any R-net JSM, including LCD based versions, as well as an Omni or any other Input Device.



The LED JSMs can actually be used with any R-net PM, ISM or any other Output Device.



The SM can only be used with the EL PM90.




The JSM-Led should not be used for Latched drive operation, there is no facility to fit an E-stop switch to the LED Joystick Module.

SK7798I/5

## 2 Controls – JSM-LED and JSM-LED-L

There are common controls between the R-net and R-net LED control systems that operate as previously described in Chapter 1. Where a control has been moved this will be noted, however where a control has changed, a description of its new functionality and use will be given.

All information regarding handling advice, cleaning requirements and overall control described in Chapter 1 should be followed.

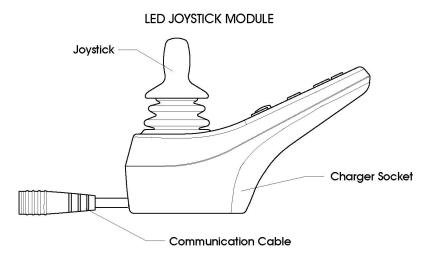


CONTROL PANEL VARIANTS
Without Lighting
With Lighting

Mode

II4 SK7798I/6

#### 2.I Buttons

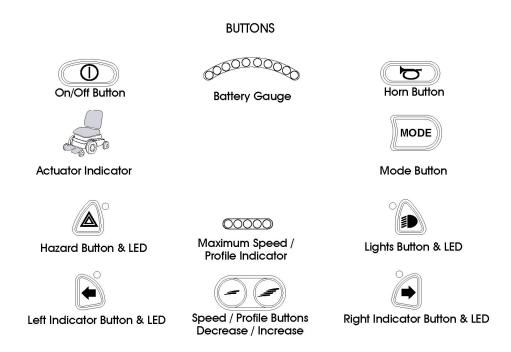

## 2.I.I Battery Gauge

The battery gauge shows you that the wheelchair is switched on. It also indicates the operating status of the wheelchair. Details are given in section 3.

If the battery gauge shows red, yellow and green, the batteries are charged. (LEDs 1-10)

If the battery gauge shows just red and yellow, then you should charge the batteries as soon as you can. (LEDs 1 - 7)

If the battery gauge shows just red, either steady or flashing slowly, then you should charge the batteries immediately. (LEDs 1 – 3)



Do not operate the control system if the battery is nearly discharged. Failure to comply with this condition may leave the user stranded in an unsafe position, such as in the middle of a road. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

## 2.1.2 Maximum Speed / Profile Indicator

This is a gauge which shows the maximum speed setting for the wheelchair or, if the control system is programmed for drive profile operation, the selected drive profile.

This gauge also indicates if the speed of the wheelchair is being limited or if the control system is locked, refer to section 3.

## 2.I.2.I Maximum Speed Indicator

This is a gauge that shows the maximum speed setting of the wheelchair. There are five speed settings – step 1 is the lowest speed and step 5 is the highest speed. For details of how to change the maximum speed setting, see sections 2.1.3 and 2.1.4.

#### 2.1.2.2 Profile Indicator

This is an indicator that shows the selected drive profile. There may be up to 5 drive profiles available, this depends on the programming of the control system. For details of how to select drive profiles, see sections 2.1.3 and 2.1.4.



A Control System utilizing an LED JSM should only be programmed with a maximum of 5 Profiles enabled.

#### 2.1.3 Speed / Profile Decrease Button

This button decreases the maximum speed setting or, if the control system is programmed for drive profile operation, selects a lower drive profile.

It is possible to program the control system so this button has no effect while the wheelchair is being driven.

Refer to Chapter 3 - Programming for details.

#### 2.1.4 Speed / Profile Increase Button

This button increases the maximum speed setting or, if the control system is programmed for drive profile operation, selects a higher drive profile.

It is possible to program the control system so this button has no effect while the wheelchair is being driven.

Refer to Chapter 3 - Programming for details.

#### 2.I.5 Mod∈ Button

The Mode button allows the user to navigate through the available operating Modes for the control system. The available modes are dependent on programming and the range of auxiliary output devices connected to the control system.

Refer to Chapter 3 - Programming for details.



When in any Mode other than Drive and Seating the Speed and Actuator LEDs are all extinguished.

## 2.2 Actuator Indicator

This LED set displays which Actuator channel is currently being controlled when the Control System is in Actuator Mode. The actuators can be programmed to work in multiple ways, refer to Chapter 3 Programming for details.

Actuator selection and operation is achieved using the Joystick.

Motions to the Left or Right select different actuator channels.

Motions Forward and Backwards move the actuator(s) selected.

II6 SK7798I/6

## 3 Control System Status indication

The battery gauge and maximum speed /profile indicator show the status of the control system.

A number of supposedly defective control systems returned to us are subsequently found to operate correctly. This indicates that many reported faults are due to wheelchair problems rather than the control system.

#### 3.1 Battery Gauge is Steady

This indicates that all is well.

#### 3.2 Battery Gauge Flashes Slowly

The control system is functioning correctly, but you should charge the battery as soon as possible.

## 3.3 Battery Gauge Steps Up

The wheelchair batteries are being charged. You will not be able to drive the wheelchair until the charger is disconnected and you have switched the control system off and on again.

#### 3.4 Battery Gauge Flashes Rapidly (even with the joystick released)

The control system safety circuits have operated and the control system has been prevented from moving the wheelchair.

This indicates a system trip, i.e. the R-net has detected a problem somewhere in the wheelchair's electrical system. Please follow this procedure:

- Switch off the control system.
- Make sure that all connectors on the wheelchair and the control system are mated securely.
- Check the condition of the battery.
- If you can't find the problem, try using the self-help guide given in section 3.7.
- Switch on the control system again and try to drive the wheelchair. If the safety circuits operate again, switch off and do not try to use the wheelchair.

Contact your service agent.

## 3.5 Speed Indicator Ripples Outwards

In this instance the LEDs make a ripple motion starting with the middle LED and then stepping outwards on both sides. The Control System has detected that a new module has been added and is reconfiguring.

#### 3.6 Speed Indicator LEDs 2 & 4 Flash

**Speed 2 + 4** When the control system requires a reboot; for example, after a module re-configuration, the second and fourth speed indicator LEDs will flash.



### 3.7 Self-Help Guide

If a system trip occurs, you can find out what has happened by counting the number of LEDs on the battery gauge that are flashing.

Below is a list of self-help actions. Try to use this list before you contact your service agent. Go to the number in the list which matches the number of flashing LEDs and follow the instructions.

If the problem persists after you have made the checks described below contact your service agent.

-	,
1 LED	The battery needs charging or there is a bad connection to the battery. Check the connections to the battery. If the connections are good, try charging the battery.
2 LED	The left hand motor* has a bad connection. Check the connections to the left hand motor.
Z LED	The left hand thold. That a bad confidential, check the confidential of the left hand thold.
••	
3 LED	The left hand motor* has a short circuit to a battery connection. Contact your service agent.
4.1.	
4 LED	The right hand motor* has a bad connection. Check the connections to the right hand motor.
5 LED	The right hand motor* has a short circuit to a battery connection. Contact your service agent.
6 LED	The wheelchair is being prevented from driving by an external signal. The exact cause will depend on the type of wheelchair you have.
7 LED	A joystick fault is indicated. Make sure that the joystick is in the center position before switching on the control system.
8 LED	A possible control system fault is indicated. Make sure that all connections are secure.
9 LED	The parking brakes have a bad connection. Check the parking brake and motor connections. Make sure the control system connections are secure.
10 LED	An excessive voltage has been applied to the control system. This is usually caused by a poor battery connection. Check the battery connections.
7 LED+ S	A communication fault is indicated. Make sure that the joystick cable is securely connected and not damaged.
Actuator Flash	An Actuator trip is indicated. If more than one actuator is fitted, check which actuator is not working correctly. Check the actuator wiring.

<sup>\*</sup> If the programmable parameter, Motor Swap has been enabled, then left and right hand references in this table will need transposing.

## 3.8 Slow or Sluggish Movement

If the wheelchair does not travel at full speed or does not respond quickly enough, and the battery condition is good, check the maximum speed setting. If adjusting the speed setting does not remedy the problem then there may be a non-hazardous fault. Contact your service agent.

## 3.9 Maximum Speed / Profile Indicator is Steady

The display will vary slightly depending on whether the control system is programmed to operate with drive profiles. For more information on drive profiles, refer to Chapter 3.

II8 SK7798I/6

#### 3.10.1 Speed Indication

The number of LEDs illuminated shows the maximum speed setting. For example, if the setting is speed level 4, then the four left hand LEDs will be illuminated.

#### 3.IO.2 Profile Indication

The LED illuminated shows the selected drive profile. For example, if drive profile 4 is selected, then the fourth LEDs from the left will be illuminated.

#### 3.II Maximum Speed / Profile Indicator Ripples Up and Down

This indicates the control system is locked, refer to Chapter 1 section 5.4 for details of how to unlock the control system.

#### 3.12 Maximum Speed / Profile Indicator Flashes

This indicates the speed of the wheelchair is being limited for safety reasons. The exact reason will depend on the type of wheelchair, however, the most common cause is that the seat is in the elevated position.

## 4 Getting Ready to Drive

- Operate the On/Off switch. The battery gauge will blink then remain on after a few seconds.
- Check that the maximum speed control is set to a level that suits you.




If you push the joystick before or just after you switch the control system on, the battery gauge will ripple up and down. You must release and center the joystick to resume normal operation. If you do not release the joystick within five seconds the wheelchair will not be able to move, even if you release the joystick and push it again. The battery gauge will flash 7 LEDs at this time. You can reset this condition by switching the control system off and on again. Refer to section 3.

If the battery gauge flashes rapidly, then the R-net has detected a problem somewhere in the wheelchair's electrical system. Refer to section 3 for details.

# 5 Mounting and Installation

The modules that make up the LED range should be mounted as per the instructions in Chapter 2. Although the overall dimensions are different, in the case of the power module and the joystick module, the mounting hole size and positioning is the same.

Connecting the modules together is possible using the standard R-net Communication Cables, except in the case of the Seating Module that utilizes a specific cable that is supplied with the module.

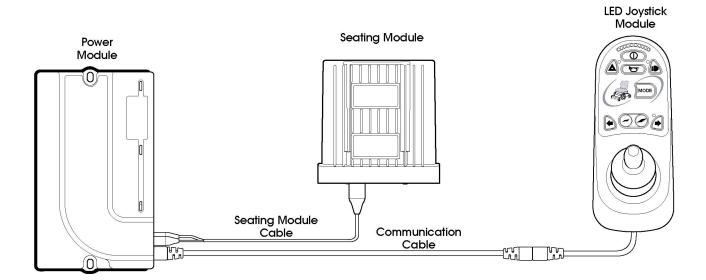



The Seating Module is connected to the EL Power Module using the Seating Module Cable, which connects to the Inhibit 2 and Actuator Channel I connections on the Power Module.

I2O SK7798I/6

## 6 Power Module & SM Wiring

#### 6.I General

Study the data sheets for the EL Power Module and the Seating Module to identify:

- The output current, ratings and restrictions.
- The connector pin assignments.

Recommendations for the cross-sectional area, ratings and materials for wiring are given in the table in Chapter 2. These depend on the application. You are responsible for establishing the suitability of the particular wiring arrangement used on the wheelchair. PGDT can make general recommendations for wiring to R-net control systems, but PGDT accepts no responsibility for the wiring arrangement used.

Make sure that the connectors you use are reliable under all operating conditions and correctly wired with no short circuits. Do not use unsuitable components - it may result in poor wheelchair reliability.



The chair manufacturer is responsible for establishing the suitability of the particular wiring arrangements used on the wheelchair, for both normal use and stalled conditions. PGDT can make general recommendations for wiring for R-net control systems, but PGDT accepts no responsibility for, and accepts no liability for losses of any kind arising from, the actual wiring arrangement used.



The chair manufacturer is responsible for ensuring that only the mating connectors specified by PGDT on the control systems' specific data sheet are used to connect to the control system. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.




The chair manufacturer is responsible for ensuring that suitable connectors are used and securely mated throughout the chair wiring system and that the workmanship associated with the wiring system is of a good enough quality. Failure to meet this condition could result in intermittent operation, sudden stopping or veering, or even create a burn or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.


The information pertaining to connectors, crimps and crimp tool part numbers can be found in Chapter 2.

## 6.I.I Power Module Wiring

The following diagram gives details of the LED Power Module connections

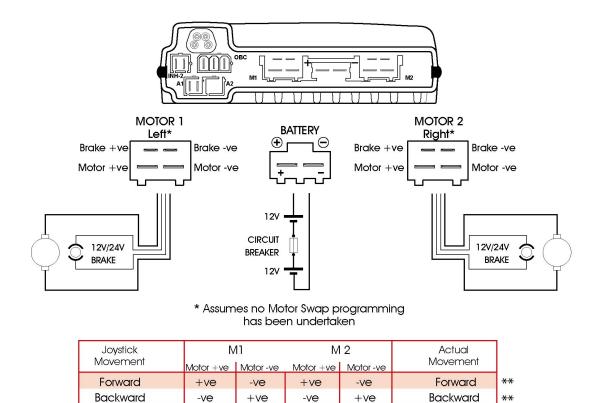


\*\* Assumes no Joystick Orientation, Invert M1 Direction or Invert M2 Direction programming has been undertaken

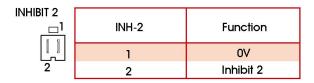

	Joystick Movement	Pin 1	Pin 2	Actuator Movement	Actuator				
I	Forward	-ve	+ve	Channel Up	*				
ı	Backward	+ve	-ve	Channel Down	* 2				
ľ	* Assumes no lovetick Orientation								

\* Assumes no Joystick Orientation programming has beeen undertaken

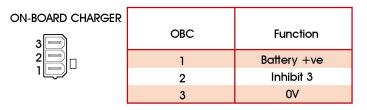


The Power Module is shipped with rubber bungs inserted into some of the connectors. The A2 connector is blank and covered with a rubber bung. This should not be removed. Only remove the rubber bung from the required connectors.

I22 SK7798I/6

#### 6.1.2 Seating Module Wiring

The following diagram gives details of the Seating Module connections.

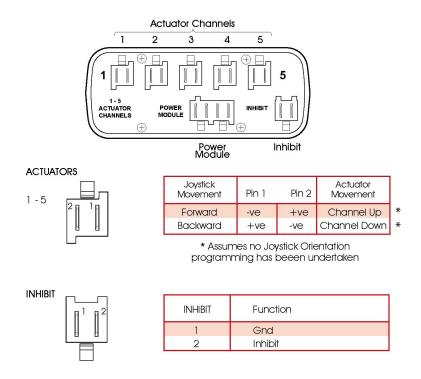

## 6.1.3 Crimping

Refer to Chapter 2- Installation.

## 6.2 Wire Gauge and Types

Refer to Chapter 2- Installation.

## 6.3 Battery and Motor Connections

Refer to Chapter 2- Installation.

## 6.4 EL PM to Seating Module Connections

Use only a PGDT supplied Seating Module Cable to connect between the EL PM and the Seating Module. The 4-way connector on this cable carries Actuator and Inhibit 2 connections, while the two 2-way connectors on the other end should be connected to the EL PM's Actuator and Inhibit 2 connections.



If it is required to connect a switch directly to the Inhibit 2 connector on the EL PM, then the 2-way connector for Inhibit 2 on the Seating Module Cable should not be connected.

#### 6.5 Inhibit Connections

As with standard PMs, the EL PM90 contains the Inhibit 2 and 3 connections. However, if the EL PM90 is used in conjunction with a Seating Module, then Inhibit 2 can be routed through to that module via the Seating Module Cable.

The Inhibit connections can be used to limit speed, inhibit drive and /or inhibit actuator channels. Refer to Chapter 3 – Programming for full details.

## 6.6 Actuator Connections

The EL PM90 has one actuator channel, which can be used in single actuator applications. However, the connection of a Seating Module consumes this channel, but effectively splits it into a maximum of 5. All actuator channels have a maximum rating of 12A.



The Seating Module does not support automatic end-stop detection. It is therefore recommended that actuators with series end-stop switches are used.

124 SK77981/6

## 7 Programming

This section describes the programming functions that are associated with the Seating Module.



For any of the following programming to be effective, the parameter Seating Module Enabled must be set to Yes.

#### :: Actuator Axes Explained ::

An actuator axis means a particular function of seat control. An axis may involve just one actuator motor, for example, a simple seat lift; or an axis may involve multiple motors, for example, adjustment of both footrests simultaneously or an anti-shear recline system. There are 5 actuator motor channels on the SM and a maximum of 12 axes are available. Unlike the ISM, the SM's 12 axes are pre-defined in terms of which channels are driven. This is shown in the table below.

Channels		Axis										
	1	2	3	4	5	6	7	8	9	10	11	12
1	Х						Х			Х	Х	Х
2		x					х	х			х	х
3			Х					Х	Х			Х
4				х		х			х	х	х	Х
5					Х	Х				Х	Х	Х

For example: when Axis 1 is selected, then Channel 1 will be driven; and when Axis 6 is selected, then both Channels 4 and 5 will be driven.

For an axis to be available to the user, it must be enabled. The relevant programmable parameter is Axis Enabled.

The R-net SM can support up to 12 actuator axes with a maximum current of 12A across any combination of actuator channels.

The user display for a particular axis is programmable. The relevant parameter is Axis Display.

## 7.1 Seating Module Enabled

This parameter sets whether a Power Module is compatible with an ISM or with a Seating Module.

The parameter options are Yes or No.

If set to Yes, the Power Module will be compatible with a Seating Module. This setting should also be used if it is intended to use an EL PM90 to drive a single actuator.


If set to No, the Power Module will be compatible with an ISM.

#### Seating Module - Global

## 7.2 Actuator Mode Entry Axis - (Act. Entry Axis)

This sets the actuator axis that is selected when Seating Mode is entered.

The programmable options are in the form of a drop-down menu, as shown in the following diagram.

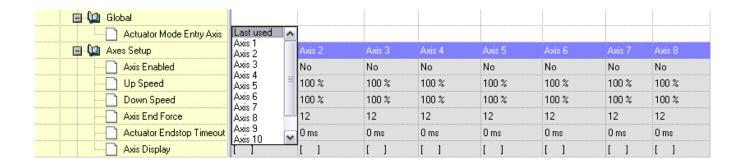

Simply select the required axis. If Last used is selected, this means the axis that was selected on the last occasion Seating Mode was used will be the selected axis when Seating Mode is re-entered.

## Seating Module - Axes Setup

#### 7.3 Axis Enabled

This parameter sets if an axis is available to a user.

The parameter options are Yes or No.

If set to Yes, the axis is available.

If set to No. the axis is not available.

#### 7.4 Up Speed - (UpSpd)

This sets the speed of the SM channel(s) in a given axis in the Up direction.

Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward.

Programmable range is 25% to 100% in steps of 1%.

#### 7.5 Down Speed - (DnSpd)

This sets the speed of the SM channel(s) in a given axis in the Down direction.

Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced in reverse.

Programmable range is 25% to 100% in steps of 1%.

## 7.6.I Axis End Force

This parameter sets the maximum force for a particular actuator Axis. The parameter is programmable between 2-12 in steps of 1. The PG default value is 12. The value of Endforce roughly equates to Amps.

#### 7.7 Actuator Current Limit Timeout

This parameter sets the time after which Endforce will not be applied to the actuator. It is programmable between 0 (OFF) and 5000ms in steps of 10ms. The PG default value is 0 (OFF).


#### 7.6 Axis Display

This parameter sets which Actuator Indicator LED(s) on a JSM-LED are illuminated when a particular axis is selected.


If the Seating Module is being used in conjunction with an LCD based Module, then this parameter will set which elements in the wheelchair symbol on the display are illuminated.

I26 SK7798I/6


The programmable options are in the form of a drop down menu, as shown in the following diagram.

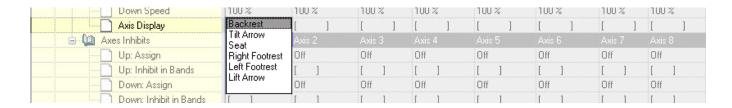

In the case of the JSM-LED, the programmable options relate to the wheelchair symbol as below.

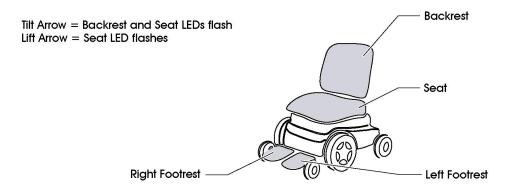

In the case of an LCD based Joystick Module, the programmable options relate to the wheelchair symbol as below.

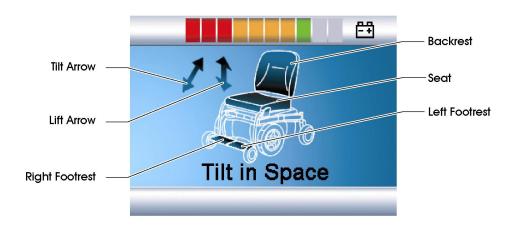


#### :: Example ::

To configure the R-net to control actuator motors connected to the left and right footrests simultaneously, the previously described parameters should be set as follows.

Axis 6 Enabled – Yes

Axis 6 Display - 4,5 (Left Footrest, Right Footrest)

Actuator Mode Entry Axis	Last used							
🖨 🛍 Axes Setup	Axis 1					Axis 6	Axis 7	
	No	No	No	No	No	Yes	No	No
Up Speed	100 %	100 %	100 %	100 %	100 %	100%	100 %	100 %
Down Speed	100 %	100 %	100 %	100 %	100 %	100%	100 %	100%
Axis Display	[ ]	[ ]	[ ]	[ ]	[ ]	[ 45]	[ ]	[ ]
🖨 🙋 Axes Inhibits								
Lin: Assign	Off	O#	Off	Off	O#	O#	Off	Off

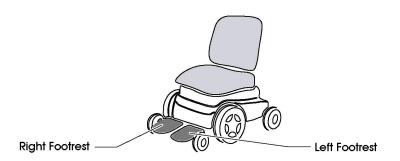

In this instance, selecting Axis 6 on the Joystick Module will illuminate both Footrest LEDs, and movement of the joystick either forward or backwards will activate the actuator motors connected to channels 4 and 5.

#### Seating Module - Axes Inhibits

Each actuator axis has a software inhibit for its up direction. Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward. The inhibit can be initiated by any actual hardware inhibit in the system.

The parameter, Assign, links the software inhibit to an actual hardware inhibit. For example, if Assign: is set to 2, then the axis can be inhibited by Inhibit Input 2.

The programmable options are Off and 2 to 33.

If set to Off, then the relevant Axis x Up Inhibit can never be active.


If set to a numeric value, then the relevant Axis x Up Inhibit will be under the control of the Inhibit Input that corresponds to that number.

# 7.78 Axis x - Up Inhibit in Bands (x = I to G)

This sets the condition of the input (assigned to Axis x Up Inhibit) that will result in an inhibit of the actuator drive in the up direction. Up is defined as pin 2 positive with respect to pin 1 when the joystick is displaced forward. The inhibit can be set to occur in any combination of bands. See *Inhibits Explained* in Chapter 3 Programming for more details on bands.

The programmable options are Band 0, Band 1, Band 2 and Band 3.

If the parameter is selected, then a drop down menu as below will appear and the bands required to invoke the inhibit should be selected. For example, if the inhibit is required for Band 3 only, then that option should be selected, as shown.

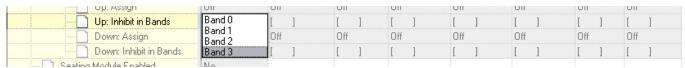

When the drop down menu is closed, a 3 will be shown in the cell.

128 SK77981/6

## 7.9 Axis x - Down Assign(x = I to I2)

Each actuator axis has a software inhibit for its down direction. Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced in reverse. The inhibit can be initiated by any actual hardware inhibit in the system.

The parameter, Assign, links the software inhibit to an actual hardware inhibit. For example, if Assign: is set to 2, then the axis can be inhibited by Inhibit Input 2.

The programmable options are Off and 2 to 33.

If set to Off, then the relevant Axis x Down Inhibit can never be active.

If set to a numeric value, then the relevant Axis x Down Inhibit will be under the control of the Inhibit Input that corresponds to that number.

# 7.10 Axis x - Down: Inhibit in Bands (x = 1 to 6)

This sets the condition of the input (assigned to Axis x Down Inhibit) that will result in an inhibit of the actuator drive in the down direction. Down is defined as pin 1 positive with respect to pin 2 when the joystick is displaced backwards. The inhibit can be set to occur in any combination of bands. See *Inhibits Explained* in Chapter 3 Programming for more details on bands.

The programmable options are Band 0, Band 1, Band 2 and Band 3.

If the parameter is selected, then a drop down menu as below will appear and the bands required to invoke the inhibit should be selected. For example, if the inhibit is required for Band 3 only, then that option should be selected, as shown.



When the drop down menu is closed, a 3 will be shown in the cell.

I30 SK7798I/6



# **CHAPTER 5 - ENCODER MODULE**

B2 SK7798I/6

## I Introduction

The Encoder Module is used on wheelchair with encoders fitted to the motors.


An encoder is a device that monitors the speed and direction of the wheelchair motors. The encoder converts information such as position and direction of rotation of the motor into a digital signal which can be used by the R-net system to accurately control the speed and direction of the wheelchair.



The PGDT Encoder Module should only be used with 24V Encoders, with a current rating of 75mA per channel.

## 2 Installation


The Encoder Module can connect into the R-net serial bus at any convenient point.

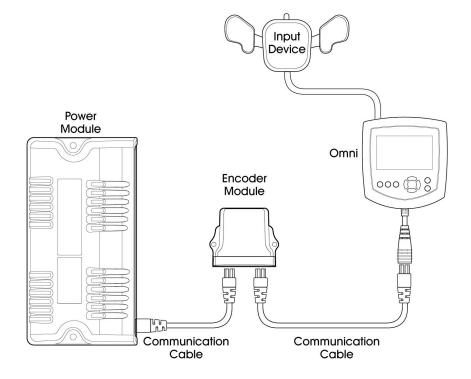

# 3 Mounting

#### 3.I Orientation

The Encoder Module should be mounted so that water cannot enter and remain in the connector recesses. The recommended mounting orientation is such that the Module is lying flat. The function of the Encoder Module is not sensitive to mounting orientation. The electronics compartment of the controller has an IPX4 dust and water rating.

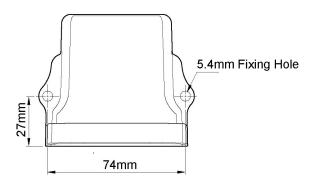

#### 3.2 Position


The Encoder Module must be mounted in a position where it is not exposed to levels of water, dust, shock or vibration above those expected on a mobility application. The Encoder Module has been tested in accordance with ISO7176/14 with respect to these conditions. Contact PGDT if you need further advice.


## 3.3 Connections

There are two 4-way connectors, one for each encoder. The pin-out for these connectors is as below.

Encoder Module connector kit is D51261 or the parts can be sourced directly from Molex

Housing- 43025-0400 Crimp- 43030-0007

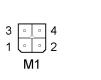

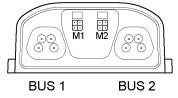



M1/M2 Pin	Function
Supply +24V	1
Encoder Ground	2
Encoder A	3
Encoder B	4


 $\ensuremath{\mathsf{M1}}$  and  $\ensuremath{\mathsf{M2}}$  pin-out are viewing rear of mating connector.

For Encoder A and Encoder B signal details, refer to the sections Invert M1 and Invert M2.

B4 SK7798I/6

## 4 Programming

Within the Encoders branch of the PC Programmer, there are the following parameters.

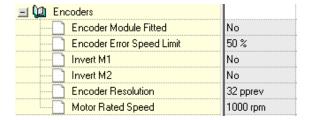

Additionally, to ensure system safety should an encoder become defective or disconnected, it is also necessary to set the standard R-net parameter, Compensation.

Each of the parameters are described below.

#### 4.I Encoder Module Fitted

This sets whether the Power Module checks for the presence of an Encoder Module, which in turn checks for correctly functioning encoders.

The programmable options are Yes and No.

If set to Yes and functioning encoders are not connected, then an error condition will occur.

If set to  ${\hbox{No}}$  and functioning encoder are not connected, then conventional drive control will occur.

## 4.2 Encoder Error Speed Limit

This sets the maximum speed of the wheelchair should an encoder error be detected.

The programmable range is 0% to 100% in steps of 1%.

## 4.3 Invert MI

This reverses the way the speed and direction signals are read from the encoder fitted to the motor that is connected to the Power Module's M1 connector.

The programmable options are Yes and No. The table below shows the effect of each option.

Motor Direction	Invert M1	Polarity of Encoder B on a rising edge of Encoder A
Forward	No	Low
Reverse	No	High
Forward	Yes	High
Reverse	Yes	Low

#### 4.4 Invert M2

This reverses the way the speed and direction signals are read from the encoder fitted to the motor that is connected to the Power Module's M1 connector.

The programmable options are Yes and No. The table below shows the effect of each option.

Motor Direction	Invert M2	Polarity of Encoder B on a rising edge of Encoder A	
Forward	No	Low	
Reverse	No	High	
Forward	Yes	High	
Reverse	Yes	Low	

#### 4.5 Encoder Resolution

This must be set to the number of pulses per revolution that the encoders output.

The programmable range is 1pprev to 128pprev in steps of 1pprev.

#### 4.6 Motor Rated Speed

This should be set to the motor manufacturer's stated speed for the motor at 24V. The R-net's speed control loop will then give a full forward motor speed as expressed below, assuming the parameter, Forward Speed, is set to 100% and no speed limits are active.

Full forward motor speed = (21.6/24) x Motor Rated Speed

The programmable range is 150rpm to 9775rpm in steps of 1rpm



If a slower full forward motor speed is required, then the parameter, Forward Speed, should be set accordingly.

#### 4.7 Compensation

Even though this parameter is not used for speed control, to ensure safety, particularly if an encoder becomes defective or disconnected, then it is still essential this value is set correctly.

The programmable range is 0mOhms to 500mOhms in steps of 5mOhms.

PGDT recommend this value is initially set to 70% of the total resistance of the motor's armature, and all the connector and cables between the motor and the Power Module motor output. Motor manufacturers should be able to supply a figure the motor's armature resistance, and typical connector and cables may be around 40mOhm.

Example: The motor has a specified armature resistance of 100mOhms, and cables and connectors between the motor and the Power Module are estimated to be 40mOhm. This gives a total values of 140mOhm. The value of Compensation should therefore be set to 70% of 140mOhm = 100mOhm.

B6 SK7798I/6



# **CHAPTER 6 - DIAGNOSTICS**

I38 SK7798I/6

## I Introduction

The primary objective of this chapter is to assist service personnel in finding the likely area of a detected trip within the whole wheelchair electrical system. It is important to realize that even though the control system is signalling a trip, it may not be the control system itself that is defective. This is because the control system is able to detect problems in other electrical components (motors, batteries, solenoid brakes, etc.) or, more importantly, the wiring to them. When a control system has detected a trip, a system trip is indicated.

This chapter covers the diagnostic process when working with an LCD based JSM or a DTT. If just a JSM-LED is being used, refer to section Control System Status Indication in Chapter 4.



Diagnostics should only be conducted by healthcare professionals with in-depth knowledge of PGDT electronic control systems. An incorrect or badly effected repair could result in an unsafe set-up of a wheelchair. PGDT accepts no liability for losses of any kind arising from an incorrect or badly effected repair.

#### I.I Diagnostic Procedure

To diagnose a trip please follow this procedure:

- Read and note the Trip Text displayed, the Identified Module and the Trip Code. Refer to section 2.
- Switch off the control system.
- Make sure that all connectors on the listed Module and the wheelchair are mated securely.
- Check the condition of the battery.
- Find the definition of the Trip Text, and take the required action. Refer to section 3.
- Switch on the control system again and try to drive the wheelchair. If the safety circuits operate again, switch off
  and do not try to use the wheelchair.
- Contact your service agent.


## 2 Diagnostic Screens

## 2.1 Current Diagnostic Screen

When the control system safety circuits have operated and the control system has been prevented from moving the wheelchair a diagnostics screen will be displayed.

This indicates a system trip, i.e. the R-net has detected a problem somewhere in the wheelchair's electrical system.

If the error is in a non-active module, for example in the ISM but with a Drive Mode selected, then drive will still be possible, however, the diagnostic screen will appear intermittently.

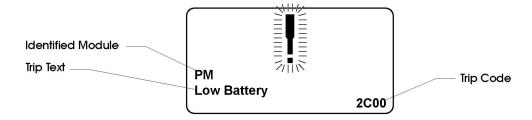

#### 2.I.I Identified Module

This identifies which module of the control system has registered the problem, such:

PM Power Module

JSM Joystick Module

ISM Intelligent Seating/lighting Module

#### 2.1.2 Trip Text

The Trip Text gives a brief description of the trip type.

## 2.I.3 Trip Cod€

The 4 digit code displayed gives the exact trip that has been recorded.

#### 2.1.4 Example

The example screen shown in section 2.1 shows the following information

Identified Module Power Module
Trip Text Low Battery
Trip Code 2C00

This means the battery needs charging or there is a bad connection to the battery.

Check the connections to the battery. If the connections are good, try charging the battery.

#### 2.2 System Log Screen

All trips are stored in the system log, whether they have been resolved or are still active. The system log stores the Trip Text and the number of occurrences. The trips are stored in their respective Modules within the system. The system log can be accessed using OBP, a Diagnostic Test Tool or the R-net PC Programmer.

Entering the System Log using OBP.

- Select System from the menu.
- Select Diagnostics from the menu.
- This displays the Diagnostic screen that shows the connected modules and the version history. See diagram below.
- If a Module has had no trips then the message No Entries will be displayed, otherwise something similar to the following screen will be displayed.

		Diagnostics
JSM	1.9	
ΡМ	1.9	
ISM	1.9	

I40 SK7798I/6

Inhibit


Inhibit 2

Power Module

## 2.3 Connections

Inhibit 2

The R-net has various connections that are referred to by the diagnostic text. These connections can be on the Power Modules(PM + ELPM), Joystick Modules(JSM + JSM LED) and the Seating Modules(LM, SM + ISM). The following diagram shows the connections on these Modules.

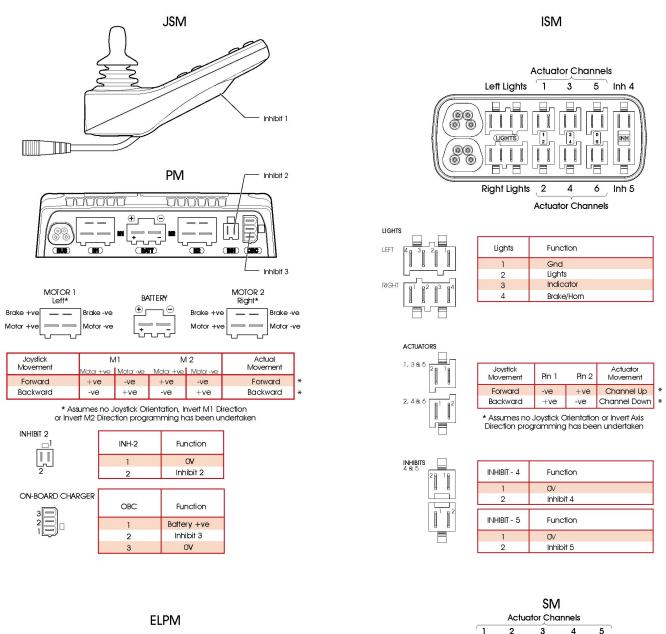



The R-net has 5 Inhibit connections within these 3 Modules, but programming allows for multiple inhibit connections to be added in further PGDT and 3rd party Modules. For details of these inhibit connections refer to the Wheelchair Manufacturers documentation.

# 3 Diagnostic Text Definitions

Once a trip text and module have been established use the following definitions to ascertain the possible cause and required corrective procedure.

Trip Text	Referral Point
Center Joystick	Refer to section 3.1
Joystick Error	Refer to section 3.1
Low Battery	Refer to section 3.2
High Battery	Refer to section 3.3
M1 Brake Error	Refer to section 3.4
M2 Brake Error	Refer to section 3.4
M1 Motor Error	Refer to section 3.5
M2 Motor Error	Refer to section 3.5
Inhibit Active	Refer to section 3.6
Jstick Cal Error	Refer to section 3.7
Brake Lamp Short	Refer to section 3.8
Left Lamp Short	Refer to section 3.9
Right Lamp Short	Refer to section 3.9
L Ind Lamp Short	Refer to section 3.10
R Ind Lamp Short	Refer to section 3.10
L Ind Lamp Failed	Refer to section 3.11
R Ind Lamp Failed	Refer to section 3.11
Over-current	Refer to section 3.12
Overtemp. (Acts)	Refer to section 3.13
Overtemp. (Lamps)	Refer to section 3.14
DIME Error	Refer to section 3.15
Memory Error	Refer to section 3.16
PM Memory Error	Refer to section 3.17
Bad Cable	Refer to section 3.18
Bad Settings	Refer to section 3.19
Module Error	Refer to section 3.20
System Error	Refer to section 3.21
SID Detached	Refer to section 3.22
Switch Detached	Refer to section 3.23
Gone to Sleep	Refer to section 3.24
Charging	Refer to section 3.25
M1 Sensor Error	Refer to section 3.26
M2 Sensor Error	Refer to section 3.26
EM disconnected	Refer to section 3.26

## 3.1 Center Joystick

The most common cause of this trip is if the joystick is deflected away from center before and during the time the control system is switched on. The joystick displaced screen will be displayed for 5 seconds, if the joystick is not released within that time then a trip is registered.

• Ensure that the joystick is centered and power-up the control system.

If the trip is still present then the joystick or Joystick Module may be defective.

142 SK7798I/6



Depending on the Wheelchair Manufacturer's service agreement, the joystick may be replaced as a separate part. Otherwise refer to section 5.

#### 3.2 Low Battery

This occurs when the control system detects that the battery voltage has fallen below 16V.

Check the condition of the batteries and the connections to the control system.

If the trip is still present after the batteries and connections have been checked, then the Power Module may be defective. Refer to section 5.

#### 3.3 High Battery Voltage

This occurs when the control system detects that the battery voltage has risen above 35V. The most common reasons for this are overcharging of the battery or bad connections between the control system and the batteries.

• Check the condition of the batteries and the connections to the control system.

If the trip is still present after the batteries and connections have been checked, then the Power Module may be defective. Refer to Section 5.

#### 3.4 Brake Error

This occurs when the control system detects a problem in the solenoid brakes or the connections to them.

- 1505 M1 Brake Error Refer to section 2.3 for connection details
- 1506 M2 Brake Error Refer to section 2.3 for connection details
- Check the solenoid brakes, cables and connections to the control system.

If the trip is still present after the above checks have been made, then the Power Module may be defective. Refer to Section 5.

#### 3.5 Motor Error

This occurs when the control system detects that a motor has become disconnected.

- 3B00 M1 Motor Error Refer to section 2.3 for connection details
- 3C00 M2 Motor Error Refer to section 2.3 for connection details
- Check the motors, cables and connections to the control system.

If the trip is still present after the above checks have been made, then the Power Module may be defective. Refer to Section 6.

#### 3.6 Inhibit Active

This occurs when any of the Inhibit inputs are active and in a latched state.

The actual inhibit that is active is indicated by the last 2 digits in the Trip Code. The code is in Hex.

- 1E01 For Inhibit 1 Off-Board Charger is connected. Refer to section 2.3 for connection details.
- 1E20 For Inhibit 2 Refer to section 2.3 for connection details
- 1E21 For Inhibit 3 Refer to section 2.3 for connection details
- 1E22 For Inhibit 4 Refer to section 2.3 for connection details
- 1E23 For Inhibit 5 Refer to section 2.3 for connection details
- Cycle the power. This will drop out of Latched Mode that might clear the trip.
- Check all wiring and switches connected to the indicated Inhibits.

If the trip is still present after the above checks have been made, then the PM or ISM may be defective. Refer to Section 5.

#### 3.7 Joystick Calibration Error

This occurs when the Joystick Calibration process has not been successful.

• Enter OBP and attempt calibration.

If the trip is still present after the above has been attempted, then the Joystick Module may be defective. Refer to Section 5.

#### 3.8 Brake Lamp Short

This occurs when the control system detects a short in the Brake Lamp Circuit. Refer to section 2.3 for connection details.

Check the brake lamps, cables and connections to the control system.

If the trip is still present after the above checks have been made, then the ISM may be defective. Refer to Section 5.

#### 3.9 Lamp Short

This occurs when the control system detects a short in either of the Lamp Circuits. Refer to section 2.3 for connection details.

- 7205 Left Lamp Short
- 7209 Right Lamp Short
- Check the lamps, cables and connections to the control system.

If the trip is still present after the above checks have been made, then the ISM may be defective. Refer to Section 5.

## 3.10 Indicator Lamp Short

This occurs when the control system detects a short in either of the Indicator Circuits. Refer to section 2.3 for connection details

- 7206 Left Indicator Short
- 720A Right Indicator Short
- Check the indicators, cables and connections to the control system.

If the trip is still present after the above checks have been made, then the ISM may be defective. Refer to Section 5.

#### 3.II Indicator Lamp Failed

This occurs when the control system detects a failure in either of the Indicator Circuits. This is most likely to be an indicator bulb failure. Refer to section 2.3 for connection details

- 7207 Left Indicator Failed
- 7208 Right Indicator Failed
- Check the indicator bulbs, cables and connections to the control system.

If the trip is still present after the above checks have been made, then the ISM may be defective. Refer to Section 5.

#### 3.12 Over-current

This occurs when the control system detects an excessive amount of current in an Actuator Channel.

This may be due to a faulty endstop switch, actuator motor, cables or connections.

- Check the movement of the actuator is not obstructed.
- Check the endstop switches (if fitted) are terminating the power to the actuator motor.

If the trip is still present after the above checks have been made, then the SM or ISM may be defective. Refer to Section 5.

144 SK7798I/6

#### 3.I3 Overtemp (acts)

This occurs when the control system detects that the SM or ISM's actuator circuitry has become too hot. The control system will cease drive to the actuator motor in question.

- Allow the SM or ISM to cool.
- If the SM or ISM is frequently overheating check the condition of the actuator motors and the connections to them.
- If the trip persists contact your service agent.

### 3.14 Overtemp (lamps)

This occurs when the control system detects that the ISM's lighting circuitry has become too hot. The control system will cease supplying current to the lamp in question.

- Allow the ISM to cool.
- If the ISM is frequently overheating check the condition of all the connected bulbs and lamps.
- If the trip persists contact your service agent.

#### 3.I5 DIME Error

This occurs when the control system detects an identification conflict between two modules in the system.

If a new module has been introduced:

- Disconnect the new module and cycle the power.
- If no trip is present connect the new module to the system and cycle the power.
- If the trip reappears then the new module must be the cause of the problem.

If there have been no additions:

• Disconnect one module at a time and cycle the power.

If the trip is still present after the above checks have been made, contact your service agent.

## 3.16 Memory Error

This is a non-specific memory error that could be caused by any of the modules within the system.

- Check all cables and connections.
- Cycle the power.

If the trip is still present and the system contains 3rd Party Modules:

• Disconnect all the non-PGDT modules and cycle the power.

If this has cleared the trip:

- Connect each 3rd Party Module in turn, cycling the power each time.
- If the trip reappears after one of the power cycles then the last module to have been added to the system must be defective.

If the trip is still present after the above checks have been made, then the PM may be defective. Refer to Section 5.

#### 3.17 PM Memory Error

This is a specific Power Module based trip.

- Check all cables and connections.
- Using the R-net PC Programmer, re-program the control system.

This should be done with either the most current specific program file for the wheelchair or the manufacturer's original programming file.



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT control systems. Incorrect programming could result in an unsafe set-up of a wheelchair for a user. PGDT accepts no responsibility for losses of any kind if the programming of the control system is altered from the factory pre-set values.

If the trip is still present after the above checks have been made, then the PM may be defective. Refer to Section 5.

#### 3.18 Bad Cable

This occurs when the control system detects a fault in the wiring in the communication cables between any of the modules.

- Check all cables and connections for continuity.
- If there is any visible damage to cables, replace and cycle power.
- Disconnect one cable from the system at a time cycling the power after each disconnection.

If the trip is still present after the above checks have been made, then the PM may be defective. Refer to Section 5.

#### 3.19 Bad Settings

This occurs when the control system detects incorrect or invalid program settings.

- Check all parameter settings and re-program the control system using the R-net PC Programmer.
- Make a note of the current parameter settings and then reset the control system to default settings.
- Re-program the required settings in small groups, cycling the power after each group to see if the trip occurs.

If the trip is still present after the above checks have been made, then the PM may be defective. Refer to Section 5.



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT control systems. Incorrect programming could result in an unsafe set-up of a wheelchair for a user. PGDT accepts no responsibility for losses of any kind if the programming of the control system is altered from the factory pre-set values.

#### 3.20 Module Error

This occurs when the control system detects a trip within a specific module. The module will be identified on the diagnostics screen as described in section 2.

- Check all cables and connections.
- Cycle the power.

If the trip is still present after the above checks have been made, then the module identified may be defective. Refer to Section 5.

#### 3.21 System Error

This occurs when the system detects a trip that cannot be attributed to a specific module.

- Check all cables and connections.
- Cycle the power.

If the trip is still present and the system contains 3rd Party Modules:

• Disconnect all the none-PGDT modules and cycle the power.

146 SK7798I/6

If this has cleared the trip:

- Connect each 3rd Party Module in turn, cycling the power each time.
- If the trip reappears after one of the power cycles then the last module to have been added to the system must be defective.

If the trip is still present after the above checks have been made, then the PGDT control system may be defective. Refer to Section 5.

#### 3.22 SID Disconnected

The Omni has detected that the Specialty Input Device (SID) has become disconnected.

• Check all cables and connectors between the Omni and the SID.

If the error persists:

Check that the setting of the parameter, 9-Way Detect, is appropriate for the SID that is being used. For example,
if the SID has no detect-link, then this parameter should be set to Off.

If the trip is still present after the above checks have been made, then the Input Device may be defective. Contact your service agent.

#### 3.23 Switch Detached

The Omni has detected that the User Switch has become disconnected.

• Check all cables and connectors between the Omni and the User Switch.

If the trip is still present after the above checks have been made, then the User Switch may be defective. Contact your service agent.

If it is required to use the Omni without a User Switch being connected, then the parameter, Switch Detect, should be set to Off. If a User Switch is not used the responsibility for that decision lies with the healthcare professional.



Because a disconnected User Switch means there is no emergency stop function, PG Drives Technology recommend that Switch Detect for this option is always set to On. PG Drives Technology will accept no liability for any losses resulting from any other setting of this parameter.

### 3.24 Gone to Sleep

This occurs when the control system has been left inactive for a time greater than the parameter Sleep Timer.

An entry is made in the system log each time this occurs.

## 3.25 Charging

This occurs when the control system detects that a charger is connected to either Inhibit 1 or Inhibit 3. Refer to section 2.3 for connection details

The Battery charging screen will be displayed during charger connection.

An entry is made in the system log each time this occurs.

If an On-Board Charger is used:

Disconnect the charger from the AC supply.

If an Off-Board Charger is used:

• Disconnect the charger from the Wheelchair.

If the trip is still present after the charger has been disconnected then the Joystick Module may be defective. Refer to Section 5.

## 3.26 Encoder Error

This occurs when the control system detects an error with the Encoder Module or the Encoder Module has become disconnected

- 2001 M1 Sensor Error The power module cannot detect an Encoder module connection to MI
- 3C00 M2 Sensor Error The power module cannot detect an Encoder module connection to M2
- 7801 EM disconnected- The power module cannot detect an encoder module fitted when the parameter Encoder Module Fitted set to Yes
- Check the motors, cables and connections to the control system.

If the trip is still present after the above checks have been made, then the Encoder Module may be defective. Refer to Section 5.

148 SK7798I/6

## 4 Basic Tests

After a repair has been completed, the following tests should be carried out. These are minimum recommendations, depending on the nature of the original trip then additional tests may be required.



These tests are a minimum recommendation only. It is the responsibility of the service person(s) to perform other tests relevant to the original trip and wheelchair type. Refer to the wheelchair's Technical Manual for exact information of other tests. PGDT accepts no liability for losses of any kind arising from the carrying out of the described tests, or from not carrying out additional relevant tests.



These tests should be conducted in an open space and a restraining device such as a seat belt should always be used. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.

## 4.I General Inspection

Make sure all connectors are securely mated.

- Check the condition of all cables and connectors for damage.
- Check the rubber gaiter or boot around the base of the joystick shaft for damage. Check visually only, do not handle the gaiter.
- Make sure that all components of the control system are securely mounted.
- Do not overtighten any securing screws.

### 4.2 Brake Test

These tests should be carried out on a level floor with at least one meter clear space around the wheelchair.

- Switch on the control system.
- Check the screen remains on after initialisation.
- Push the joystick slowly forwards until you hear the parking brakes operate. The wheelchair may start to move.
- Immediately release the joystick. You must be able to hear each parking brake operate within 2 seconds.
- Repeat the test a further three times, pushing the joystick slowly backwards, left and right.

## 4.3 Drive Test

With the maximum speed control in the minimum position, drive the wheelchair in all directions, ensuring the drive is comfortable and easy to control for the user.

Repeat the above but with the speed control set to maximum.

#### 4.4 Gradient Test



Before carrying out this test ensure another person is present to prevent the wheelchair from tipping backwards.

Drive the wheelchair forwards up its maximum rated gradient. While on the gradient release the joystick and ensure the wheelchair comes to rest and the brakes are applied without the front wheels lifting of the ground.

Deflect the joystick forwards and continue driving up the slope. Ensure the pick up is smooth and positive.

Stop the wheelchair and reverse down the gradient. While on the gradient release the joystick and ensure the wheelchair comes to rest and the brakes are applied without the front wheels lifting off the ground.

## 4.5 Lights, Indicators and Hazard Lamps Test

If lights are fitted:

- Visually check each bulb for correct illumination.
- Check each indicator bulb for correct illumination and that the flash rate is 1.5Hz ± 0.5Hz.
- Disconnect each bulb in turn and check that the remaining bulb for that side flashes at  $3Hz \pm 0.5Hz$ .

If hazard lamps are fitted:

Check each bulb for correct illumination and that the flash rate is 1.5Hz ± 0.5Hz.

#### 4.6 Actuator Test

If actuators are fitted:

- Check each motor for correct direction of movement.
- Ensure the mechanical end-stops are secure and that they stall the actuator motors, thus operating the ISM's automatic end-stop detection.

## 4.7 Inhibit Input Test

Insert an appropriate battery charger, or equivalent inhibit connection, into the Charger Connector on the JSM and ensure that drive is inhibited.

If Inhibits 2, 3, 4 and 5 are used for inhibit or speed limiting purposes, suitable tests should be devised to ensure correct operation.

I50 SK7798I/6

# 5 Servicing of Defective Units

Excluding specific OEM approved replacement parts (for details of these contact the wheelchair manufacturer), there are no serviceable parts in the R-net control system. Consequently, any defective units must be returned to PGDT or a PGDT approved service organization for repair.



Any replacement work carried out without the wheelchair manufacturer's permission will invalidate the control system's warranty.

Opening or making any unauthorized adjustments or modifications to the R-net control system or its components will invalidate any warranty and may result in hazards to the vehicle user, and is strictly forbidden.



PGDT accepts no liability for losses of any kind arising from unauthorized opening, adjustments or modifications to any component of the R-net control system.

I52 SK7798I/6



# **CHAPTER 7 - WARNING SUMMARY**

I54 SK7798I/6

## I Introduction

This section summarizes all of the very important warnings that appear throughout the text of this manual. Do not install, maintain or operate the machine without reading, understanding and observing the following warnings. Failure to observe these warnings could result in UNSAFE CONDITIONS for the user of a machine or affect the reliability of the control system. PG Drives Technology accepts no liability for losses of any kind arising from failure to comply with any of the conditions in the warnings listed below. Failure to observe these warnings will invalidate the R-Net warranty.

The machine manufacturer may wish to use this section as a check list, to ensure the risk areas identified below have been addressed within their own machine designs and associated documentation.

## 2 Warnings

## 2.I Charger Socket



The control system's warranty will be voided if any device other than a PG Drives Technology. Programmer, or the battery charger supplied with the wheelchair, is connected into this socket. Chapter I Section 4.4.

## 2.2 Driving Technique



The wheelchair user must be capable of driving a wheelchair safely. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter I Section 8.2.

#### 2.3 Hazards



Although the R-Net control system is designed to be extremely reliable and each unit is rigorously tested during manufacture, the possibility of a system malfunction always exists (however small the probability). Under some conditions of system malfunction the control system must (for safety reasons) stop the chair instantaneously. If there is any possibility of the user falling out of the chair as a result of a sudden braking action, it is imperative that a restraining device such as a seat belt is supplied with the wheelchair and that it is in use at all times when the wheelchair is in motion. PGDT accept no liability for losses of any kind arising from the unexpected stopping of the wheelchair, or arising from the improper use of the wheelchair or control system.



Do not operate the control system if the chair behaves erratically, or shows abnormal signs of heating, sparks or smoke. Turn the control system off at once and consult your service agent. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Electronic equipment can be affected by Electro Magnetic Interference (EMI). Such interference may be generated by radio stations, TV stations, other radio transmitters and cellular phones. If the chair exhibits erratic behavior due to EMI, turn the control system off immediately and consult your service agent. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



It is the responsibility of the chair manufacturer to ensure that the wheelchair complies with appropriate National and International EMC legislation. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The wheelchair user must comply with all wheelchair safety warnings. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter I Section 9.1.

#### 2.4 Battery Charging



Do not operate the control system if the battery is nearly discharged. Failure to comply with this condition may leave the user stranded in an unsafe position, such as in the middle of a road. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter I Section II.

## 2.5 Battery Charging



Do not exceed the maximum charging current of I2 A rms. Always use an off-board charger fitted with a Neutrik NC3MX plug. Failure to observe these conditions could result in poor contact resistance in the charger connector resulting in overheating of the charger plugs. This presents a potential burn hazard for the user. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Ensure that the charger plug pins are of the correct polarity to be compatible with the pin polarity shown on the control system's specific data sheet. Failure to observe this condition could result in a burn hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Do not disconnect batteries or open-circuit the circuit breaker while charging is in progress. Failure to observe this condition could result in a burns hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Only use the battery charger that has been supplied with your wheelchair. The use of incorrect chargers could damage the batteries, wheelchair, control system or charger itself, or may result in parts overheating creating the potential for burns or even fire. PGDT accepts no liability for losses of any kind if the charger is incompatible with the control system (see Chapter 2, section 7) or any other part of the wheelchair system. Chapter I Section I2.

#### 2.6 Programming



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT electronic control systems. Incorrect programming could result in an unsafe set-up of a wheelchair for a user. PGDT accepts no liability for losses of any kind if the programming of the control system is altered from factory pre-set values. Chapter I Section I4.

## 2.7 Joystick Knobs



Do not replace the joystick knob with any unauthorized item It may cause hazardous operation. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter I Section IS.

#### 2.8 Servicing



PGDT accept no liability for losses of any kind arising from unauthorized opening, adjustment or modifications to the R-Net control system.



If the control system is damaged in any way, or if internal damage may have occurred through impact or dropping, have the product checked by qualified personnel before operating. PGDT

I56 SK7798I/6

accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter I section I6.

## 2.9 Warranty



The warranty will be void if the R-Net has not been used in accordance with R-Net Technical Manual SK7798I, the R-Net has been subject to misuse or abuse, or if the R-Net has been modified or repaired by unauthorized persons. Chapter I Section I7.

## 2.10 Program Settings



It is the manufacturer's responsibility to program the control system to suit the vehicle model and ensure safe operation in compliance with relevant legal requirements over the whole of the operating range. PGDT accepts no liability for losses of any kind due to failure to, or incorrect programming or the R-net Control System. Refer to Chapter 3 for programming details.



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT electronic control systems. Incorrect programming could result in an unsafe setup of a wheelchair for the user. PG Drives Technology accepts no liability for losses of any kind if the programming of the control system is altered from factory pre-set values. PGDT accepts no liability for losses of any kind if the drive or stability characteristics of the chair are altered without prior notification and discussion with PGDT. Chapter 2 section I.2.

#### 2.II Charger Interlock



The chair manufacturer is responsible for providing a means of preventing the use of the wheelchair while the batteries are being charged. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 2 Section 2.2.

#### 2.12 Power Module Wiring - General



The chair manufacturer is responsible for establishing the suitability of the particular wiring arrangements used on the wheelchair, for both normal use and stalled conditions. PGDT can make general recommendations for wiring for R-Net control systems, but PGDT accepts no responsibility for, and accepts no liability for losses of any kind arising from, the actual wiring arrangement used.



The chair manufacturer is responsible for ensuring that only the mating connectors specified by PGDT on the control system's specific data sheet are used to connect to the control system. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The chair manufacturer is responsible for ensuring that suitable connectors are used and securely mated throughout the chair wiring system and that the workmanship associated with the wiring system is of a good enough quality. Failure to meet this condition could result in intermittent operation, sudden stopping or veering, or even create a burn or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 2 Section 6.1.

#### 2.13 Crimping



Defective or poor quality crimps may affect the warranty of the control system. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 2 Section 6.I.3.

#### 2.14 Wire Gauge and Types



Battery and motor wires should have IO5°C rated PVC insulation.



The chair manufacturer is responsible for establishing the suitability of the particular wiring arrangements used on the wheelchair. PGDT can make general recommendations for wiring for R-net control systems, but PGDT accepts no responsibility for, and accepts no liability for losses of any kind arising from, the actual wiring arrangement used. Chapter 2 Section 6.2.

#### 2.15 Battery Connections



The chair manufacturer must install a suitable circuit breaker to provide protection against short circuits in the battery wiring, power loom or the control system. Failure to comply with this could result in a fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 2 Section 6.3.

#### 2.16 Battery Charging



Do not exceed the maximum charging current of I2A rms. Always use a charger fitted with a Neutrik NC3MX plug. Failure to observe these conditions could result in poor contact resistance in the charger connector resulting in overheating of the charger plugs. This presents a potential burn hazard for the user. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Ensure that the charger plug pins are of the correct polarity to be compatible with the pin polarity shown on the control system's specific data sheet. Failure to observe this condition could result in a burn hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Do not disconnect batteries or open-circuit the circuit breaker while charging is in progress. Failure to observe this condition could result in a burns hazard or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 2 Section 7.1.

## 2.17 Drive Motors



The chair manufacturer is responsible for ensuring that the control system is matched to the motor resistance. Failure to do this may result in poor control characteristics, which in extreme instances can make a chair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The chair manufacturer is responsible for always ensuring that any replacement motors or gearboxes are fully compatible with the originals that the control system was designed to match. Failure to do this may result in poor control characteristics, which in extreme instances can make

I58 SK7798I/6

a chair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



Users must not move a control system from one chair type to install it on a different chair type. Control systems with different part numbers may have both hardware and software differences to ensure that they are compatible with the electrical and dynamic characteristics of their specific target vehicles. The characteristics of one type of control system may not be the compatible with a different, unauthorized chair. Failure to observe this warning could result in an unsafe setup for the wheelchair user and may create a fire hazard depending on the motors, wiring, connectors and circuit breakers installed on the unauthorized chair. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 2 section 8.

#### 2.18 Solenoid Brakes



It is essential that the control system is programmed for the correct brake voltage. If it is not, damage may occur to the brakes or the drive performance of the wheelchair may be affected. Chapter 2 section 9.

## 2.19 Introduction



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT control systems. Incorrect programming could result in an unsafe set-up of a wheelchair for a user. PGDT accept no responsibility for losses of any kind if the programming of the control system is altered from the factory pre-set values. Chapter 3 Section I.

## 2.20 How to Enter OBP Mode



If the keycode method is set, then it may be possible for a wheelchair user to gain entry to the OBP function. PGDT accept no liability for losses of any kind if this method of OBP is set. PGDT also accept no liability for losses of any kind if OBP access is gained by unauthorised personnel obtaining a dongle. Chapter 3 Section 2.1.

## 2.21 Tremor Damping



If using high values of Tremor Damping, pay particular attention to the stopping distance of the wheelchair, as it will be increased. Chapter 3 Section 6.2l.

## 2.22 Emergency Stop Switch



PGDT accept no liability for losses of any kind resulting from the use of a wheelchair with latched drive or actuator operation and with no normally closed Emergency Stop Switch fitted. Chapter 3 Section 7.10.

#### 2.23 OBP Keycode Entry



If this parameter is set to Yes, then it may be possible for a wheelchair user to gain entry to the OBP function. PGDT accept no liability for losses of any kind if this method of OBP is set. PGDT also accept no liability for losses of any kind if OBP access is gained by unauthorised personnel obtaining a dongle. Chapter 3 Section 7.II.

#### 2.24 Latched Drive



PGDT accept no liability for losses of any kind resulting from the use of a wheelchair with latched drive or actuator operation and with no normally closed Emergency Stop Switch fitted. Chapter 3 Section 8.1 & 8.2.

## 2.25 Latched Timeout



PGDT accept no liability for losses of any kind resulting from the use of a wheelchair with latched drive or actuator operation and with a Latched Timeout value of less than 2. Chapter 3 Section 8.3.

## 2.26 Actuator Acceleration



This parameter must be set to IOO% on systems that use automatic endstops, i.e. those that do not use micro-switches at the end of actuator travel. If the value is not set to IOO%, then the automatic endstop function will not operate and damage to the actuator motor or the wheelchair frame may occur. Chapter 3 Section 9.4.

#### 2.27 Actuator Deceleration



This parameter must be set to IOO% on systems that use automatic endstops, i.e. those that do not use micro-switches at the end of actuator travel. If the value is not set to IOO%, then the automatic endstop function will not operate and damage to the actuator motor or the wheelchair frame may occur. Chapter 3 Section 9.5.

## 2.28 Brake Fault Detect



This parameter should only ever be set to No if there are no electrical brakes fitted to the wheelchair. Chapter 3 Section II.2.

## 2.29 Brake Voltage



It is essential the correct value for the brakes' operating voltage is set. An incorrect value could result in damage to the brakes or motors. PGDT accept no liability for losses of any kind resulting from incorrect programming of this parameter. Chapter 3 Section II.3.

### 2.30 Soft Stop Rate



It is the wheelchair manufacturer's responsibility to ensure that the wheelchair complies with the correct legislation for braking distances for the country in which the wheelchair is to be sold. PGDT accept no liability for losses of any kind resulting from incorrect programming of this parameter. Chapter 3 Section II.4.

## 2.31 Maximum Current Limit, Boost Drive Current & Boost Drive Time



The values in the table above should never be exceeded. Doing so will invalidate the warranty and affect the long term reliability of the Power Module. Chapter 3 Section I2.I.

I60 SK7798I/6

## 2.32 Compensation



These tests should be conducted in an open space and a restraining device such as a set belt should always be used. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The wheelchair manufacturer is responsible for ensuring that the value of Compensation is correctly set for the motors being used on the wheelchair. This is particularly important if replacement motors are being sent out to upgrade a wheelchair already in the field. Failure to comply with this requirement may result in poor control characteristics, which in extreme instances can make a wheelchair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The wheelchair manufacturer is responsible for always ensuring that any replacement motors supplied as spare parts are fully compatible with the originals that the system was programmed to match. Failure to comply with this requirement may result in poor control characteristics, which in extreme instances can make a wheelchair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



A control system must not be moved from one wheelchair type to another without first ensuring that the system is correctly programmed for the wheelchair it is intended to fit it to. Even if control systems have the same PGDT part number, they may not be programmed the same. Failure to ensure the system is correctly programmed for the wheelchair may result in a hazardous condition. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 3 Section I2.5.

#### 2.33 Compensation Factor



Inappropriate values of Compensation and Compensation Factor could make the wheelchair uncontrollable and potentially unsafe. PGDT accepts no liability for losses of any kind resulting from inappropriate setting of either or both of these parameters. Chapter 3 Section I2.II.

## 2.34 OEM Factory



It is the responsibility of the vehicle manufacturer, to ensure that adequate testing of the vehicle is carried out, to ensure that the values set will maintain vehicle stability and user safety. PGDT accepts no liability for losses of any kind resulting from this parameter being incorrectly programmed. Chapter 3 Section 15.

### 2.35 Battery Gauge



Do not operate the control system if the battery is nearly discharged. Failure to comply with this condition may leave the user stranded in an unsafe position, such as in the middle of a road. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 4 Section 2.I.I.

## 2.36 Getting Ready to Drive



If you push the joystick before or just after you switch the control system on, the battery gauge will ripple up and down. You must release and center the joystick to resume normal operation. If you do not release the joystick within five seconds the wheelchair will not be able to move, even if you release the joystick and push it again. The battery gauge will flash 7 LEDs at this time. You

can reset this condition by switching the control system off and on again. Refer to section 3. Chapter 4 section 4.

#### 2.37 General



The chair manufacturer is responsible for establishing the suitability of the particular wiring arrangements used on the wheelchair, for both normal use and stalled conditions. PGDT can make general recommendations for wiring for R-net control systems, but PGDT accepts no responsibility for, and accepts no liability for losses of any kind arising from, the actual wiring arrangement used.



The chair manufacturer is responsible for ensuring that only the mating connectors specified by PGDT on the control systems specific data sheet are used to connect to the control system. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition.



The chair manufacturer is responsible for ensuring that suitable connectors are used and securely mated throughout the chair wiring system and that the workmanship associated with the wiring system is of a good enough quality. Failure to meet this condition could result in intermittent operation, sudden stopping or veering, or even create a burn or fire hazard. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 4 section 6.

#### 2.38 Introduction



Diagnostics should only be conducted by healthcare professionals with in-depth knowledge of PGDT electronic control systems. An incorrect or badly effected repair could result in an unsafe set-up of a wheelchair. PGDT accept no liability for losses of any kind arising from an incorrect or badly effected repair. Chapter 5 Section I.

## 2.39 PM Memory Error



Programming should only be conducted by healthcare professionals with in-depth knowledge of PGDT control systems. Incorrect programming could result in an unsafe set-up of a wheelchair for a user. PGDT accept no responsibility for losses of any kind if the programming of the control system is altered from the factory preset values. Chapter 5 Section 3.17.

#### 2.40 Basic Tests



These tests are a minimum recommendation only. It is the responsibility of the service person(s) to perform other tests relevant to the original trip and wheelchair type.. Refer to the wheelchair's Technical Manual for exact information of other tests. PGDT accept no liability for losses of any kind arising from the carrying out of the described tests, or from not carrying out additional relevant tests.



These tests should be conducted in an open space and a restraining device such as a seat belt should always be used. PGDT accepts no liability for losses of any kind arising from failure to comply with this condition. Chapter 5 Section 4.

162 SK77981/6

## 2.41 Servicing of Defective Units



Any replacement work carried out without the wheelchair manufacturer's permission will invalidate the control system's warranty.



PGDT accept no liability for losses of any kind arising from unauthorized opening, adjustments or modifications to a any component of the R-net control system. Chapter 5 Section 5.

164 SK7798I/6



# **CHAPTER 8 - SPECIFICATIONS**

166 SK7798I/6

## I Electrical Specifications

Supply Voltage: 24Vdc

Operating Voltage: 16Vdc to 35Vdc

Peak Voltage:35VdcReverse Battery Protection:-40Vdc

PWM Frequency:  $20 \text{kHz} \pm 0.5\%$  Brake Voltage: 12/24 Vdc Brake Current:  $200 \mu \text{A} \text{ min.}$ 

1A max.

Charger Connector: Use only Neutrik NC3MX

**Battery Charging Current:** 12Arms max.

Maximum Drive Current:

R-Net 80 80A R-Net 90 90A R-Net 120 120A

Indicator Outputs42W per sideLighting Outputs21W per sideBrake Light Output42W total

Actuator Current:

- ISM 15A max at reduced speed. 12A max at full speed.

**- SM** 12A max.

Moisture Resistance: Electronics to IPX4

Non LCD Modules Modules with LCD Screens

Operating Temperature:  $-25^{\circ}\text{C}$  to  $+50^{\circ}\text{C}$   $-10^{\circ}\text{C}$  to  $+50^{\circ}\text{C}$ Storage Temperature:  $-40^{\circ}\text{C}$  to  $+65^{\circ}\text{C}$   $-20^{\circ}\text{C}$  to  $+65^{\circ}\text{C}$ 

EMC tested on sample wheelchair:

Susceptibility: Tested at 30V/m to EN12184 and ANSI/ RESNA requirements

Emissions: To EN55022 Class B

ESD: IEC801 part 2

I68 SK7798I/6