

User manual

American English



F5 Corpus VS

Dear Permobil User

We congratulate you on your choice of power wheelchair. Our goal is for you to continue to feel satisfied with your choice of both vendor and product. Your power wheelchair is designed to provide the highest possible comfort and safety while meeting both safety and environmental requirements.

Before you begin using your wheelchair, it is important that you read and understand the contents of these operating instructions, and in particular the safety instructions.

Produced and published by Permobil

Edition: 4

Date: 2019-09-24

Order no: 335340 eng-US

How to contact Permobil

Permobil Inc.
300 Duke Drive
Lebanon, TN 37090
USA

Tel.: 800.736.0925
Fax: 800.231.3256
E-mail: TechSupport@permobil.com
Web: www.permobilus.com

Head office of the Permobil group

Permobil AB
Box 120
861 23 Timrå
Sweden

Tel.: +46 60 59 59 00
Fax: +46 60 57 52 50
E-mail: info@permobil.com
Web: www.permobil.com

1	Usage of the standing function	11
1.1	When to use the standing function	11
1.2	When not to use the standing function	13
2	Benefits of daily standing	15
3	Important information	17
3.1	Warranty	18
3.2	Product approval	19
3.3	Technical support	19
3.4	Cyber security	20
3.5	Electromagnetic interference	20
3.6	Reporting incidents	21
3.7	Spare parts and accessories	21
3.8	Ordering documentation	22
3.9	Scrapping and recycling	22
4	Safety instructions	23
4.1	Descriptions of admonitions	24
4.2	Warnings and precautions	27
5	Getting to know your wheelchair	43
5.1	Overview	45
5.2	Technical specifications	46
5.3	Design and function	51

5.4	Control Panel	63
5.5	Accessories	126
5.6	Stickers	150
6	Preparing for use	155
6.1	Settings and adjustments	156
6.2	Handling and driving	185
6.3	Batteries	201
6.4	Transporting the wheelchair	207
7	Maintenance and repairs	217
7.1	Frequency of maintenance and inspection	221
7.2	Tool bag	222
7.3	Cleaning	223
7.4	Battery replacement	225
7.5	Batteries storage	240
7.6	Wheels and tires	242
7.7	Main circuit breaker	264
7.8	Brake release	265
7.9	Positioning belt	265
7.10	Serial number labels	266
8	Restrictions and limitations	269
8.1	Support wheels	270
8.2	Speed restriction conditions	270

8.3	Conditions that limit seat movement	273
9	Troubleshooting	275
9.1	Troubleshooting guide	276
9.2	Diagnostics R-net LCD control panel.....	277
9.3	Diagnostics R-net LED control panel	280
9.4	Repairing defective units	285
10	Electromagnetic compatibility	287
10.1	Electromagnetic emissions	288
10.2	Electromagnetic immunity	289
10.3	Recommended separation distance.....	293
	Index	294

1 Usage of the standing function

This information is based on general observations. The conditions for regular standing vary from person to person and any usage should be agreed upon with a doctor or therapist.

Only a doctor or therapist can decide which method and safe standing training can be achieved with the Permobil wheelchair.

1.1 When to use the standing function

Permobil wheelchairs with Standing function are specifically developed for people who have lost to their ability to stand (e.g. paraplegia, multiple sclerosis, cerebral palsy etc.). Standing is used for therapy and prophylaxis of:

- **Osteoporosis**

Without regular standing vertical stress of the bones, bone density can steadily decrease. This leads to the risk of bone fractures and other complications.

- **Decubitus prophylaxis**

Whilst standing the pressure on the ischial tuberosity is reduced to a minimum.

- **Skeletal misalignments and/or back pain**

Whilst seated the spinal column is naturally elongated (kyphosis; hunchback). Standing facilitates a natural spinal position and therefore better upper body stability. This prevents the risk of a scoliosis.

- **Muscular atrophy**

Without regular leg movement, the risk of muscle shortening increases (muscular atrophy) and consequently contractures may develop. Regular, self-standing training helps as movement therapy, thereby avoiding contractures and preventing possible associated pains and expensive operations.

- **Tonus regulation of spasticity**

Frequent standing helps to reduce spasticity and facilitates seat movement and positioning in the wheelchair or bed. This supports a peaceful sleep.

- **Deeper respiration**

Straightening the upper body through standing decreases the pressure of the abdominal cavity on the lungs. This improves and deepens respiration, facilitates breathing and can also help reduce the risk of pneumonia.

- **Gastrointestinal complications**

As with breathing, standing also reduces the pressure on the digestive system. This facilitates digestion (intestinal peristalsis) and helps with the improvement of bowel and bladder functions.

1.2 When not to use the standing function



WARNING!

A wheelchair with standing function can only be used upon prescription by a doctor.

Here are the most common reasons not to use the standing function:

- **Reduced bone density**

Should the bones already be very weak it is not recommended to stand. This could lead to fractures. In this case consult your doctor for a bone density measurement.

- **Poor circulation**

For heart or circulatory diseases standing should only occur under medical supervision (cardio training). Please consult a doctor in this case.

- **Severe mobility limitations or severe contractures**

2 Benefits of daily standing

The use of any stand-up device should be done only under the prescription and supervision of a medical professional. At the outset, it is recommended that your introduction to regular standing be closely monitored by your physical or occupational therapist.

It has been documented that the regular and cyclical activity of going from the seated to the standing position may offer many benefits to those that are no longer able to stand on their own. The benefits are two fold: 1) there is the ability to once again function in daily activities that necessitate standing (reaching file cabinets and equipment in the workplace, accessing cupboards/stoves/shelving at home, as well as the ability to interact eye to eye); and, 2) the potential of physical benefits that result from repeated standing (these benefits may include improved; range of motion, bone density, circulation, bowel and bladder function, etc.).

3 Important information

Before you begin using your wheelchair, it is important that you read and understand the content of these operating instructions, and in particular the safety instructions.

These operating instructions are primarily intended to acquaint you with the functions and characteristics of the wheelchair and how to use them in the best manner possible. They also contain important safety and maintenance information, as well as describing possible problems that can arise while using the wheelchair.

Always keep operating instructions within convenient reach when using your product, as the need for important information may arise concerning its use, safety and maintenance.

It is also possible to obtain information concerning our products from our website: www.permobil.com.

All information, pictures, illustrations and specifications are based upon the product information available at the time these operating instructions were printed. Pictures and illustrations used in these operating instructions are representative examples and not intended to be exact depictions of the various parts of the power wheelchair.

We reserve the right to make changes to the product without prior notice.

For those who are visually impaired, this document can be downloaded at www.permobil.com. The PDF reader magnifying tool can be used to achieve the desired text and image size.

3.1 Warranty

Contact your dealer or Permobil Inc. USA for information about the warranty period for this product.

Product Warranty Information sets forth the conditions of the warranty. For further information about applicable warranties, see <https://permobilus.com/support/warranties/>.



NOTICE

Unapproved replacement of parts

If any part is replaced without approval from Permobil, the wheelchair warranty will become void. Permobil accepts no liability for any loss that occurs as a result of a control system component being opened, adjusted or modified without permission.

3.2 Product approval

This product fulfills the requirements according to:

- ANSI/AAMI ES60601-1
- AS/NZS 60335.2.29
- AS/NZS CISPR 14.1
- CAN/CSA-C22.2 No. 60601-1
- EN60601-1
- IEC 60335-2-29
- ISO 7176-21
- ISO 7176-25
- 93/42/EEC

3.3 Technical support

In the event of technical problems, contact your dealer or call Permobil Inc. USA on 1-800-736-0925.

Be prepared to provide the wheelchair serial number, located on the chassis, to ensure proper support. See 7.10 *Serial number labels*, Page 266.

3.4 Cyber security

If you discover or suspect that someone unauthorized has tampered with, removed, or replaced a device or data derived from a device, immediately contact Permobil for support.

3.5 Electromagnetic interference

EMI stands for Electromagnetic Interference, which is an electromagnetic emission that causes a disturbance in another piece of electrical equipment. EMI can be caused by anything that carries rapidly changing electrical currents, for example mobile telephones, large medical equipment, or other sources of electromagnetic radiation. Modifications to the device can make it more susceptible to EMI. There is no easy way to evaluate the effect of a modification on the overall EMI of the device.

Your wheelchair has passed the test of immunity to electromagnetic fields. As of May 1994, 20 V/m is a generally achievable and useful immunity level, and it is also the immunity level of your wheelchair in its factory configuration. However, it cannot be excluded that electromagnetic emissions can affect your wheelchair, for example in environments such as shopping malls and hospitals.

Nevertheless, it is permitted to use cordless telephones and cell phones, including hands-free devices, while operating your power wheelchair. In the unlikely event that you should experience abnormal operation of your wheelchair, stop driving immediately, turn off the wheelchair, and report the incident as soon as possible.

3.6 Reporting incidents

If an incident occurs, please contact your nearest Permobil representative. This is usually the same person you were in contact with at the time of purchase. To get in touch with your contact, use the link at www.permobil.com. Open your country page and the contact page. The page provides the necessary contact information and a document to help you provide us with the information we need to investigate the incident. Provide as much of the information as possible; it will be of great help to us.

3.7 Spare parts and accessories

Spare parts and accessories must be ordered through your dealer.

The expected service life of this product is five years.

3.8 Ordering documentation

Should you need another copy of this manual, one may be ordered from Permobil. Ask for the order number specified on the back cover.

3.9 Scrapping and recycling

Contact Permobil for information about scrapping agreements in force.

4 Safety instructions

A power wheelchair is a motorized vehicle and therefore special care must be taken when using it.

Please read and follow all instructions and warnings in all manuals supplied with your power wheelchair and its accessories. Incorrect use may both injure the user and damage the wheelchair. In order to reduce these risks, read the User's Manual and all manuals supplied carefully, in particular the safety instructions and their warning texts.

Permobil is not responsible for personal injuries or property damage resulting from any person's failure to follow the warnings and instructions in this manual. Permobil is not responsible for injuries or damage resulting from failure to exercise good judgment.

The final selection and purchasing decision about the type of power wheelchair to be used is the responsibility of the user and his or her healthcare professional. Permobil is not responsible for inappropriate selections of models, features or improper mountings on the wheelchair.

Your power wheelchair was configured specifically for your needs as prescribed by your healthcare provider. Contact your healthcare provider to have changes in seat position or other adjustments made.

It is also of the utmost importance that you devote sufficient time to getting acquainted with the various buttons, functions and steering controls; the different seat adjustment possibilities, etc. of your wheelchair and its accessories before you begin using it.

Do not undertake your own first test drive without making sure help is available in the immediate vicinity should you need it.

4.1 Descriptions of admonitions

The following admonitions describing warnings, remarks and explanatory texts are used throughout this manual to draw attention to items of significant importance to safety:



DANGER!

Danger admonition

Indicates a dangerous situation which, if not avoided, could result in death as well as serious damage to the product or other property.

**WARNING!****Warning admonition**

Indicates a hazardous situation which, if not avoided, could result in serious injury or death as well as damage to the product or other property.

**CAUTION!****Caution admonition**

Indicates a hazardous situation which, if not avoided, could result in minor or moderate injury as well as damage to the product or other property.

**NOTICE****Notice admonition**

Indicates an important but not hazardous situation which, if not avoided, could result in damage to the product or other property.

- i** Provides information about the conditions or circumstances under which the information given applies.

4.2 Warnings and precautions



CAUTION!

Operation, driving

If the wheelchair is equipped with lights, Permobil recommends their use whenever you drive near public rights of way. Use extreme caution when driving near unprotected edges, drops or on elevated surfaces. Unintended movement or excessive speed in such areas can lead to personal injury or property damage.



CAUTION!

Operation

Do not drive the wheelchair over any curbs or obstacles higher than specified in the technical specifications section of the manual. When driving over a curb or similarly elevated surface, cross it at a 90 degree angle (perpendicular). Crossing such surfaces at any other angle may result in the wheelchair's tipping.

Reduce your speed when driving on uneven terrain or soft surfaces. Never use your wheelchair on stairs or escalators. Always take the elevator.

Do not lift or move the wheelchair by any of its removable parts. Doing so could lead to personal injury and property damage, including damage to the wheelchair.

**CAUTION!****Operation, pulling and minor impacts**

Do not use the wheelchair to pull any kind of object and never hang excessive weights on the backrest. Doing so could lead to personal injury and property damage, including damage to the wheelchair.

In the event of a collision with a wall, door or other fixed object when operating the wheelchair, always make sure all parts of the wheelchair are undamaged before operating it again. Failure to do so could lead to personal injury.

**CAUTION!****Operation, adjust seating system for elevations**

Be sure to adjust the position of your seating according to the limitations applicable to the wheelchair concerned before climbing obstacles or driving on uneven surfaces or slopes.

**WARNING!****Risk of tipping over**

Do not allow the leg rest to hit the ground when climbing obstacles, driving on uneven surfaces or slopes or when a slope levels out. Make sure there is sufficient ground clearance to avoid the risk of tipping over.

**CAUTION!****Operation**

Do not let children drive the wheelchair without supervision. Do not drive the wheelchair on public streets or roads. Obey all local pedestrian rules and be aware that vehicle drivers may have difficulty seeing you.

Do not operate your wheelchair under the influence of alcohol. Consumption of alcohol may impair your ability to operate your wheelchair safely.

Some physical impairments or the use of prescription and non-prescription medication may limit your ability to operate your wheelchair safely. Be sure to consult with your physician about your physical limitations and medications.

**WARNING!****Modifications**

Do not modify your wheelchair or any of its components. Your wheelchair has been configured specifically for your needs as prescribed by your healthcare provider. Special skills, training and knowledge are needed to set up, modify and repair the wheelchair.

Initial setup and all modifications and repairs must be performed by a qualified service technician. For warranty service, contact the dealer from whom the wheelchair was purchased.

**WARNING!****Do not alter safety parameters**

The wheelchair is equipped with certain safety parameters that limit or inhibit wheelchair functions or, under certain conditions, prevent the wheelchair from being driven. Do not alter these safety parameters.

**WARNING!****Weight limitations**

The maximum user weight for your wheelchair is set forth in the technical specifications section in this User's Manual. Operation of the wheelchair by users who exceed the maximum allowable user weight can lead to personal injury and property damage, including damage to the wheelchair, as well as voiding any warranty applicable to the wheelchair.

Do not carry passengers on the wheelchair. Doing so can lead to personal injury and property damage, including damage to the wheelchair.

**CAUTION!****Prior to riding**

In some instances, including where certain medical conditions exist, users should practice operating their wheelchair under the supervision of an assistant who is familiar with the operation of the wheelchair and with the abilities and limitations of the user.

**CAUTION!****Operation - slopes**

When driving downhill, select the slowest speed and proceed with caution. Driving down a slope may shift the user's center of gravity forward. If the wheelchair rolls faster than you would like, stop the wheelchair by releasing the joystick and begin descending again more slowly.

Avoid sudden stops or starts. Stop by releasing the joystick rather than by turning the power off. Turning off power while the wheelchair is in motion will cause the wheelchair to stop suddenly. Permobil recommends the use of securely fastened positioning belts at all times.

When driving up a slope, try to keep moving at a steady speed. Stopping and starting the wheelchair while moving up a slope makes the wheelchair more difficult to control.

Do not drive up or down slopes with gradients greater than those set forth in the technical specifications section of this User's Manual. There is a risk that the wheelchair will not maneuver safely.

**WARNING!****Operation - inclines**

Do not drive the wheelchair where the sideways gradient is greater than that set forth in the technical specifications section of this User's Manual. There is a risk of tipping over.

Do not drive up or down ramps that are not equipped with proper edge protection along the sides to prevent the wheelchair from falling off the ramp.

When driving up an incline, be sure to drive the wheelchair straight up the incline (perpendicular). Driving up an incline at an angle increases the risk of tipping or falling. Use extreme caution when driving up inclines.

Do not drive up or down hazardous inclines such as a surfaces covered with snow, ice, or wet leaves, or a surface that is uneven. Also avoid driving on ramps that do not have proper edge protection.

**WARNING!****Operation - turning**

Turning your wheelchair at high speeds may cause it to topple with personal injury as a result. The risk of tipping over is increased by high turning speeds, sharp turns, uneven surfaces, abrupt changes in direction, and driving from an area of low traction (e.g. lawn) to an area of high traction (e.g. sidewalk).

To avoid tipping over, personal injury and property damage, reduce speed and turn less sharply.

**WARNING!****Operation - freewheel mode**

In order to prevent the wheelchair from rolling away, make sure it is on a level surface before releasing the brakes.

To avoid personal injury, do not use your wheelchair in freewheel mode without an attendant present. Do not attempt to put the wheelchair in freewheel mode by yourself while sitting in it.

Do not put your wheelchair in freewheel mode while on an incline. This may cause the wheelchair to roll on its own, causing injury and property damage, including damage to the wheelchair.

**CAUTION!****Driving range**

The driving range stated in the technical specifications of this manual represents the theoretical driving range when testing the wheelchair according to RESNA WC-2, Section 4. This test is performed under ideal conditions and actual driving ranges will vary depending on battery status, tire selection and driving conditions. Frequent driving on slopes, rough ground or frequently climbing curbs etc., will also reduce the driving range.

**CAUTION!****Driving in the dark**

Driving in the dark is only permissible if the wheelchair is equipped with functioning lighting at the front and rear, or as per applicable national or local traffic regulations.

**WARNING!****Passengers**

The wheelchair is not intended to carry passengers, regardless of the age of the passenger. The maximum user weight for your seating system is stated in the technical specifications section of this User's Manual. The stated user weight includes the user and any personal effects. The maximum limit may not be exceeded. The wheelchair's maneuverability and stability can be degraded as a result.

**WARNING!****Driving with seat lift, seat tilt, backrest recline**

Make absolutely sure that nothing gets stuck between the chassis and the seat when the seat lift and/or seat tilt is operated. Operating the seat lift, seat tilt and/or backrest recline changes the center of gravity and increases the risk of tipping over. Always drive in low speed and only use these seat functions on level ground – not on hills, ramps, slopes or other inclines. Using these seat functions while driving on inclines may lead to personal injury and property damage, including damage to the wheelchair.

**WARNING!****Center of balance**

The likelihood of the wheelchair's tipping and the point at which the wheelchair will tip forward, back or to the side depends on its center of balance. Note that the following factors affect the wheelchair's center of balance:

- Elevation of the seat.
- Height and angle of the seat.
- Body position or weight distribution.
- Driving on an incline such as a ramp or a hill.
- The use of a backpack or other accessories, depending on the amount of weight added.

If the wheelchair begins to move in an unexpected manner, release the joystick immediately to stop the wheelchair. Except in an emergency, do NOT use the On/Off button to stop the wheelchair, as this will cause the wheelchair to stop abruptly which may cause personal injury.

**WARNING!****Fixed seat post**

Only authorized service providers may adjust seat height.

**WARNING!****Use installed positioning aids**

The user must use and securely fasten the positioning belt and any other positioning aids on the wheelchair. Failure to properly use positioning belts and other positioning aids may cause bodily injury. If there is any sign of damage or wear on any positioning belt or aids, immediately contact Permobil for a replacement.

**WARNING!****Support wheels**

If the wheelchair is equipped with support wheels, they must always be mounted when the wheelchair is being driven.

**WARNING!****Transfer into and out of the wheelchair**

Make sure the power is turned off before getting into or out of the wheelchair and before lifting the control side armrest.

When transferring into or out of the wheelchair, every precaution must be taken to reduce the distance between the wheelchair and the place to which the user is transferring. Too great a distance may cause the user to overexert him or herself, lose balance or fall.

Permobil recommends that users transfer in the presence of or with the assistance of an attendant.

Use caution when bending or reaching.

Never use the joystick as a handhold or support.

Do not use footplates or armrests as supports when transferring into or out of the wheelchair. The footplates and armrests are not designed as load-bearing structures. Excessive force may cause them to give way, resulting in personal injury or property damage, including damage to the wheelchair.

**NOTICE****Transporting the wheelchair**

The wheelchair may only be transported in vehicles approved for this purpose. The vehicle must be suitably designed, insured and equipped to transport a person in a wheelchair. No matter how securely it is fastened in the vehicle, a wheelchair is not designed to be a car seat and cannot offer the same degree of safety as offered by standard car seats.

Before transportation, check that the wheelchair is properly secured and that both wheel locks are engaged. Secure the wheelchair front and rear to the vehicle at the tie-down points only (marked with yellow stickers). Follow the manufacturer's instructions supplied with the approved fastening straps.

Instead of fastening straps, the wheelchair may be secured with a locking system approved by Permobil for the product concerned.

For alternative vehicle securement options, please consult your dealer.

**WARNING!****Wheelchair must be facing forward**

The wheelchair is designed to be facing forward when used as a seat in a motor vehicle.

**CAUTION!****Damage during transportation**

You must inform Permobil as soon as possible after the event if the wheelchair and its accessories have suffered transportation damage, damage during driving or damage from other causes. There is a risk that the wheelchair and its accessories can no longer be used safely and securely. Contact your service provider or Permobil for further information.

**DANGER!****User transportation in wheelchair**

When the user is seated in the wheelchair during transport, the following requirements must be met:

- The vehicle must be equipped with a locking system approved by Permobil for the product concerned. For alternative vehicle securement options, please consult your dealer.
- The locking system must be dimensioned for the total weight of the wheelchair.
- The user must use a three-point seat belt attached to the vehicle.
- The wheelchair must be crash tested and approved.
- The wheelchair must be fitted with a headrest.
- Alterations or substitutions should not be made to wheelchair securement points, structural parts, frame parts or components without consulting Permobil or the dealer.
- Spill-proof sealed batteries, such as gel or AGM, must be used.
- Make sure that wheelchair components do not accidentally press down the seat belt's release button.

If these requirements are not followed, the user can sustain bodily injuries or die.

**WARNING!****Positioning belts and aids are not vehicle seatbelts**

The wheelchair positioning belt and aids are designed to properly position the user in the wheelchair for safely operating the wheelchair. The wheelchair positioning belts and aids do not provide protection in a traffic accident and do not replace a vehicle mounted seatbelt.

**WARNING!****Secure loose objects during transportation**

Auxiliary wheelchair equipment, loose or mounted on the wheelchair, must either be secured to the wheelchair or removed from it. Dismantled or movable auxiliary equipment must be properly secured in the vehicle during transit. This is to prevent loose parts or parts that may come loose from causing injury to the occupants during transit.

**CAUTION!****Driving in extreme weather conditions**

Our wheelchairs are designed to withstand most adverse weather conditions, however to minimize the risk of being caught in difficult situations, you should avoid using the wheelchair outdoors in e.g. severe cold, heavy rain or thick snow.

Also bear in mind that certain surfaces on the wheelchair can get very hot or cold in the event of prolonged exposure to intense sunlight or cold.

**WARNING!****Environmental conditions**

Protect the wheelchair from exposure to any type of moisture, including rain, snow, mud or spray. Exposure to moisture may cause the chair to short-circuit, catch fire and cause personal injury or property damage. If it has been exposed to moisture, do not operate your wheelchair until it has dried completely.

If any of the shrouds or the joystick boot has cracks or tears, they must be replaced immediately. Failure to do so may allow moisture to enter the electronics and cause personal injury or property damage, including fire.

Be extremely careful when operating your wheelchair during icy or slippery conditions. Such conditions can lower the performance and safety of your wheelchair which could lead to an accident, personal injury and property damage, including damage to the wheelchair.

Exercise extreme caution when using oxygen in close proximity to electrical circuits and other combustible materials. Contact your oxygen supplier for instruction in the use of oxygen.

**CAUTION!****Maintenance and service**

Only carry out the minor adjustments and maintenance specified in the user manual.

All other service, repairs and maintenance on Permobil products, including control system programming, must be performed by a qualified service technician authorized by Permobil. Incorrect settings may result in unsafe operation of the wheelchair and cause it to become unstable or uncontrollable. Such modifications also void the product's warranty.

**CAUTION!****Non-approved aftermarket accessories**

Do not use parts or accessories not authorized by Permobil.

The use of non-approved aftermarket accessories and parts may cause changes in the wheelchair that make it unstable or uncontrollable.

The product warranty may be voided if unauthorized parts or accessories are used.

**CAUTION!****Non-approved electrical or electronic devices**

Connecting non-approved electrical or electronic devices to the wheelchair's electrical system may cause damage to the wheelchair and make it uncontrollable or erratic. Such use may also void the warranty.

**CAUTION!****Risk of getting caught in pinch points**

The wheelchair is heavy and contains many moving parts, which means there is an ever-present risk of getting caught in pinch points.

**WARNING!****Charging batteries**

Charge batteries in a well-ventilated room, not in a wardrobe or closet. Batteries must not be charged in a bathroom or wet room. Only use chargers with a max 10 A charging current (average value). The charging current RMS value must not exceed 12 A. When the charger is connected, the wheelchair cannot, and may not, be driven.

**WARNING!****Replacing batteries and circuit breakers**

Always turn the main circuit breaker to the Off position when replacing batteries or circuit breakers.

Exercise caution when using metallic objects during work with batteries. A short-circuit can easily cause an explosion. Always use protective gloves and goggles.

**WARNING!****Safety circuits**

The products are equipped with safety circuits. Inhibit circuits prevent the wheelchair from being driven under certain conditions. Speed reduction circuits limit the wheelchair's maximum speed under certain conditions. Limit switch circuits limit the wheelchair's functions under certain conditions. Overload protection circuits switch the wheelchair off in case of an overload. If any of these circuits stops working, stop using the wheelchair immediately and consult an authorized Permobil distributor.

Any attempt to modify the safety circuits will result in unsafe wheelchair operation and may cause the chair to become unstable or uncontrollable. Such modifications may also void the wheelchair's warranty.

**CAUTION!****Recycling batteries**

Used batteries must be disposed of responsibly in accordance with local recycling regulations.

**WARNING!****Inflating tires**

Regularly check that the wheelchair's tires have the correct pressure. Incorrect tire pressure may cause stability and maneuverability to deteriorate.

**NOTICE****Changing tires**

Avoid the use of sharp-edged tools when working with tires.

**NOTICE****Storage**

Always turn off the wheelchair and its accessories when it is not in use. Always store the wheelchair so that access for unauthorized individuals is prevented.

Never store the wheelchair in a room where there is a risk of condensation (mist or moisture on surfaces) e.g. in pool areas, laundry rooms or similar.

If you are unsure how your wheelchair and its accessories should be properly stored, contact your supplier or Permobil for more information.

**WARNING!****Damage and/or malfunctions**

If you feel the wheelchair is not behaving as it should in any regard or if you suspect that something is wrong, stop driving as soon as possible, switch off the wheelchair and contact your service provider or Permobil for further information. Refer to page 5.

**WARNING!****Flame resistance**

Fabric components have been tested for flammability. Padded parts meet the requirements of EN 1021-1, EN 1021-2 and ISO 7176-16. Plastic parts meet the requirements of UL94.

**CAUTION!****EMC requirements**

The electronics in a power wheelchair can be affected by external electromagnetic fields (e. g. from cell phones). Similarly, the electronics in the wheelchair itself also emit electromagnetic fields that may affect the immediate surroundings (e.g. certain alarm systems in businesses).

The Electromagnetic Compatibility (EMC) limit values with respect to power wheelchairs are set forth in the international standard for powered wheelchairs, ISO 7176-21.

Our power wheelchairs comply with these limit values.

**WARNING!****Discontinue use following a serious accident**

Stop using your wheelchair if it has been subject to flooding, fire, dropping, crushing or any impact event including a vehicular accident. Extraordinary events like these can lead to significant damage that is impossible to detect upon inspection.

Damage caused by these types of incidents exceeds the limits covered by the manufacturer's warranty.

For further information, contact Permobil to get in contact with a technical service supervisor.

5 Getting to know your wheelchair

5.1	Overview	45
5.2	Technical specifications	46
5.2.1	Dimensions and weight	47
5.2.2	Performance	48
5.2.3	Wheels	50
5.2.4	Electronics	50
5.2.5	Batteries	50
5.2.6	Control force	50
5.2.7	Circuit breakers	50
5.3	Design and function	51
5.3.1	Seat	51
5.3.2	Chassis	58
5.4	Control Panel	63
5.4.1	Permobil Joystick Module for R-net	63
5.4.2	Joystick panel, R-net LED control panel	110
5.4.3	ICS seat control panel	117
5.4.4	Omni2	126
5.5	Accessories	126
5.5.1	ConnectMe	127

5.6	Stickers	150
5.6.1	Read the instructions.....	150
5.6.2	Main circuit breaker also battery isolator.....	151
5.6.3	Wheel locks	151
5.6.4	Tie-down point	152
5.6.5	Support wheels.....	152
5.6.6	Crush hazard	152
5.6.7	Warning	153
5.6.8	Battery connections and main circuit breaker	154

5.1 Overview

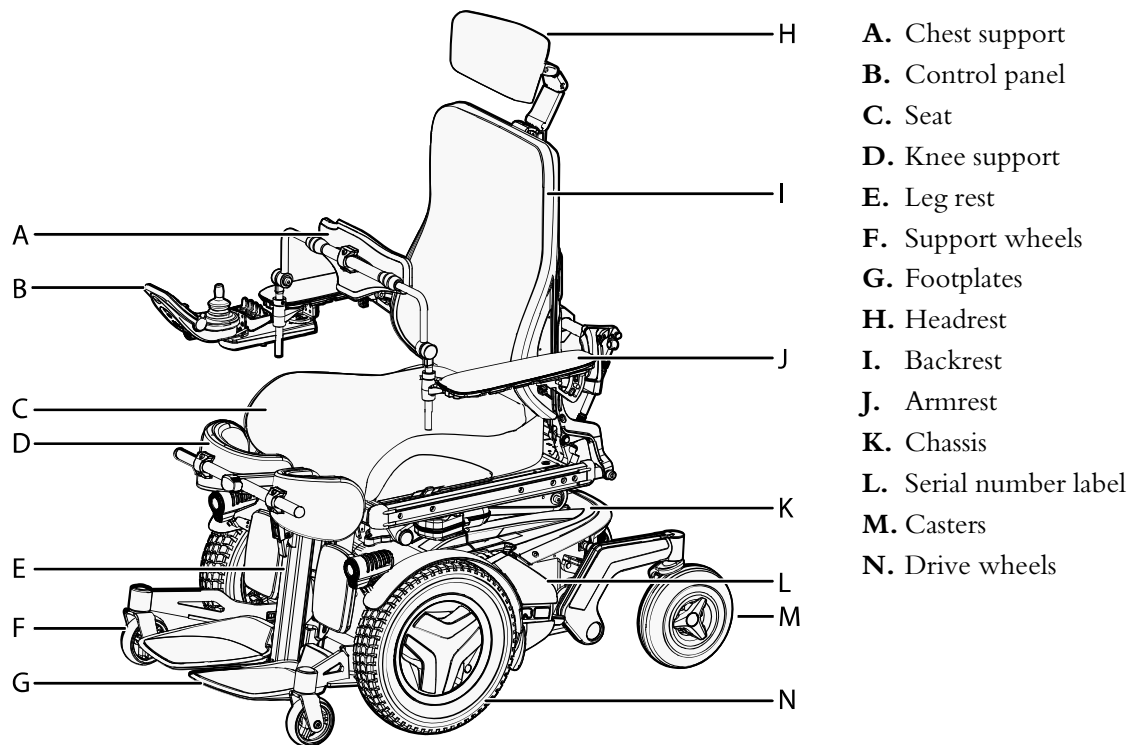


Figure 1. Overview.

The Permobil F5 Corpus VS is an electric front-wheel drive wheelchair for outdoor and indoor driving. The intended use of the F5 Corpus VS powered wheelchair is to provide indoor and outdoor mobility, including stand-up feature, to persons limited to a seating position that are capable of operating a powered wheelchair. It is intended for people with physical disabilities. Its raising function allows the user greater freedom of movement. The F5 Corpus VS can be sold over the counter or by prescription.

We recommend gradual training in standing up under the guidance of a physiotherapist before you use the Standing function every day. You will gain the greatest benefit in standing up if you regularly switch between a sitting and a standing position. This alternation is simple with the Permobil F5 Corpus VS.

The wheelchair consists of a chassis and a seat. The chassis contains the wheelchair's electronics, power supply and drive functions. The seat consists of a seat frame, seat plate/backrest, armrest/leg rest and any accessories/options such as a headrest, calf rest, chest support, etc.

5.2 Technical specifications


Product name

F5 Corpus VS

Wheelchair class

Group 4

5.2.1 Dimensions and weight

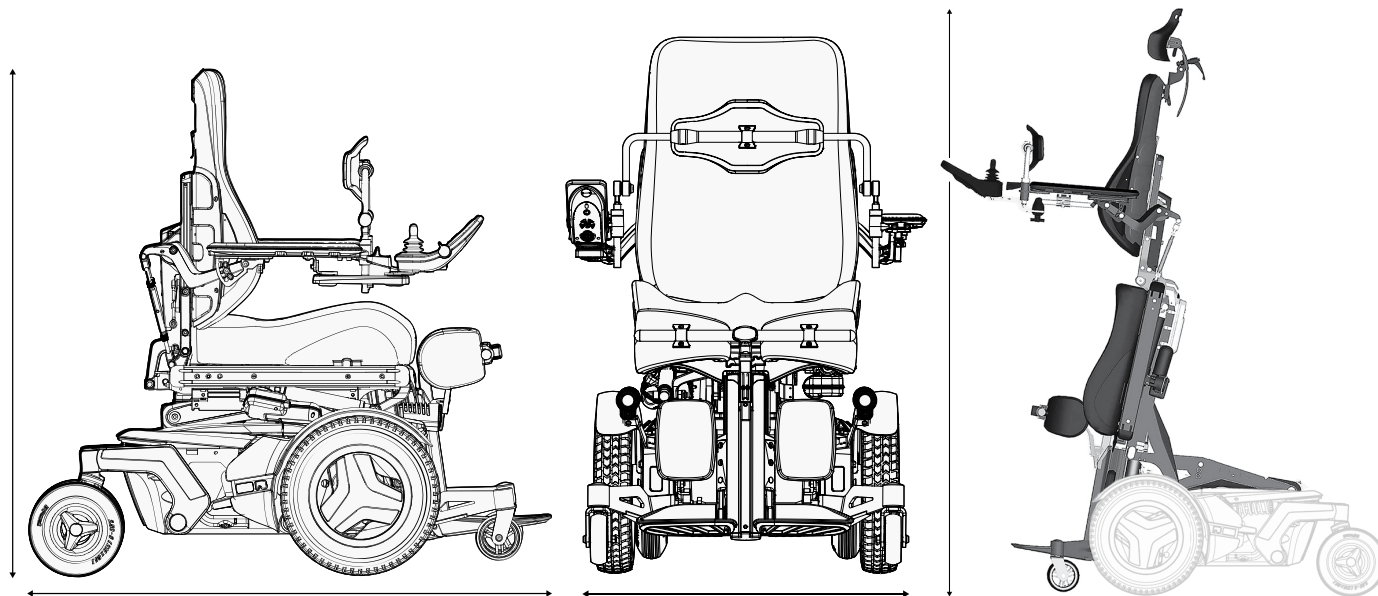

Length 46".
Height 43" – 46".

Width 25.5" – 31.0".¹

Standing position
Length 50".
Height 65" – 76".
Width 25.5" – 31.0".

1. Based on the joystick module being in the forward position.

Weight, including batteries	456 lb.
Minimum transport length	38"
Minimum transport height	32.5" – 34.5"
Minimum transport width	25.5" – 31"
Seat plane angle	-70° to 50°, -80° to 50° for users under 220 lb.
Seat depth	14" – 22" (by 1" increments)
Seat width	17" – 23" (by 2" increments)
Seat to floor height including cushion (seat surface height at front end)	19.3" – 34.6"
Seat to floor height without cushion	17.7" – 31.5"
Displacement between the wheelchair support surfaces and the user during transition from sitting to standing configuration	+0.4" along the seat plane -1.0" along the back support
Backrest angle	85° – 180°
Backrest height	19" – 24" (by 1" increments)
Backrest height without seat cushion	18.5", 21.5" – 26.5" (by 1" increments)
Footrest to seat distance	13" – 23"
Leg to seat surface angle	90° – 180°
Armrest to seat distance	7.1" – 10.2"

5.2.2 Performance

Range, sitting position	16 mi
Range, standing position	The driving range decreases by approximately 0.4 mi

Maximum speed forward, sitting position	7.5 mph
Maximum speed forward, standing position	1.6 mph
Minimum braking distance from maximum speed, sitting position	9.2 feet
Minimum braking distance from maximum speed, standing position	1.3 feet
Acceleration, standing position	4.1 ft/s ²
Ability to climb a rated slope	6°
Dynamic stability, least stable, sitting and standing position	10°
Sideways slope capability, sitting and standing position	10°
Static stability forward, sitting and standing position	12°
Static stability backward, sitting and standing position	18°
Static stability sideways, sitting and standing position	17°
Ability to negotiate obstacles, sitting position (approach distance 0")	2"
Ability to negotiate obstacles, standing position (approach distance 0")	0.7"
Ability to negotiate obstacles, sitting position (approach distance 20")	3"
Ability to negotiate obstacles, standing position (approach distance 20")	1"
Ground clearance with and without user weight	3.0", 3.1"
Maximum user weight	300 lb. (from lying to standing: 220 lb.)

Minimum turning diameter, sitting and standing position	59"
Pivot width, sitting and standing position	47"

5.2.3 Wheels

Drive wheel tire dimensions	3.00 – 8"
Caster tire dimensions	2.50 – 3" (210 x 65)
Recommended tire pressure	35 psi/250 kPa

5.2.4 Electronics

R-net drive electronics type	JSM-L-SV / PM 120
------------------------------	-------------------

5.2.5 Batteries

Recommended battery type	Lead acid, 2 x 12 V, Hybrid (AGM/GEL), group 24
Battery capacity	73 Ah

5.2.6 Control force

Force necessary to operate joystick and key pad switches	2 N
----------------------------------------------------------	-----

5.2.7 Circuit breakers

Main circuit breaker	63 A
----------------------	------

5.3 Design and function

5.3.1 Seat

5.3.1.1 Power seat functions

The power seat functions are driven by an power actuator which is steplessly controlled from the wheelchair control panel.

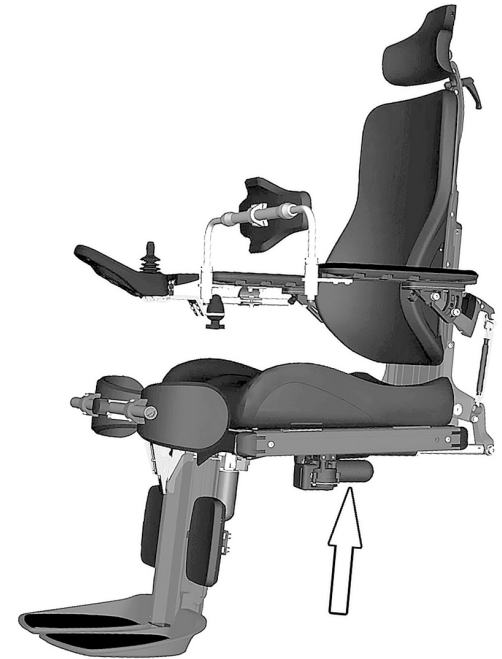


Figure 2. Power actuator.

Functions

Available functions may vary depending on the equipment fitted on your wheelchair.

- seat lift
- seat tilt
- backrest recline
- leg rest
- standing function



WARNING!

Risk of pinching while using seat functions

There is a risk of pinch-point accidents when using the power seat functions. Make absolutely sure that nothing gets stuck between moving parts. Failure to do so may lead to personal injury.

Power seat lift

A power controlled seat lift allows stepless raising or lowering of the seat in order to adjust the height to tables, benches etc.

When the seat lift is raised from the lowest position, the maximum speed of the wheelchair may be reduced.

**WARNING!****Tipping risk – driving with seat functions in use**

When operating the seat functions, the center of gravity is also shifted, increasing the risk of tipping over. Only use the seat functions on a level floor.

Always drive in the lowest speed and never tilt the seat or back so far that the wheelchair cannot be maneuvered safely.

Power seat tilt

i Applies to wheelchairs equipped with power seat tilt.



Figure 3. Power seat lift.

The power seat tilt control allows the preferred seat angle to be set within the operating range.



WARNING!

Tipping risk – driving with seat functions in use

When operating the seat functions, the center of gravity is also shifted, increasing the risk of tipping over. Only use the seat functions on a level floor.

Always drive in the lowest speed and never tilt the seat or back so far that the wheelchair cannot be maneuvered safely.

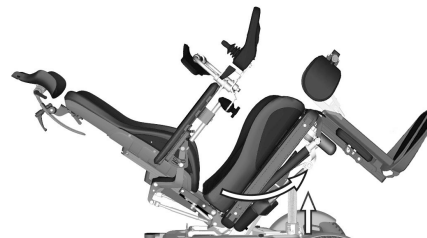


Figure 4. Power seat tilt.

Power backrest angle

The backrest angle is adjustable (via the recline control), allowing the user to set a recline angle as needed within the operating range.



WARNING!

Risk of pinching while using seat functions

There is a risk of pinch-point accidents when using the power seat functions. Make absolutely sure that nothing gets stuck between moving parts. Failure to do so may lead to personal injury.



Figure 5. Power backrest angle.

Power leg rest angle

The leg rests can be raised to the preferred angle within the operating range.



Figure 6. Power leg rests.

Standing function

The F5 Corpus VS has an electric standing function. This function makes it possible for the user to stand up in the wheelchair.



WARNING!

Risk of tipping using standing function

When the standing function is used, the center of gravity is also raised, increasing the risk of tipping over. Only use the standing function on a level floor. Never reach out from the chair. Always drive in the lowest speed.



Figure 7. Electric standing function

5.3.2 Chassis

5.3.2.1 Drive unit and shock absorbers

The wheelchair is equipped with a drive unit (B) and a shock absorber (C) for each drive wheel. The link arms are also equipped with shock absorbers (A).



NOTICE

Shock absorber adjustments should be performed by personnel who are well-acquainted with the design and functionality of the wheelchair. When adjustment is needed, contact an authorized Permobil service center.

The drive unit consists of an electric motor with a gearbox and magnetic wheel lock.

5.3.2.2 Wheels

The wheelchair's drive wheels are available with pneumatic tires or flat-free (foam filled) tires. The casters are available with solid polyurethane tires .

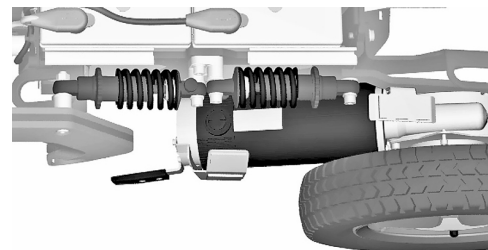


Figure 8. Drive unit and shock absorbers.

5.3.2.3 Lights and reflectors

The wheelchair is equipped with reflectors at the front and rear.

Front lights, rear lights and turn signals are optional.

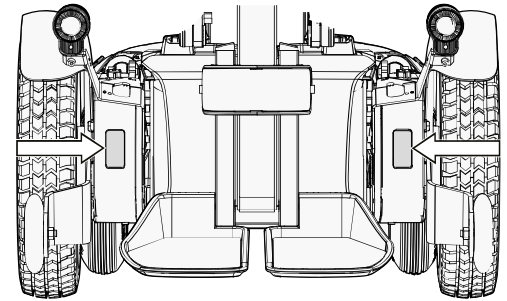


Figure 9. Front reflectors.

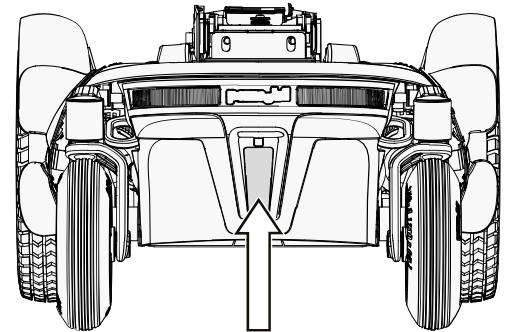


Figure 10. Back reflectors.

5.3.2.4 Batteries

The wheelchair's batteries are located under the covers of the chassis. Both batteries are easily accessible for maintenance and replacement.



WARNING!

Handling batteries - avoid short-circuit

Exercise caution when using metallic objects during work with batteries. A short-circuit can easily cause an explosion. Always use safety gloves and goggles.

Remember that the batteries are heavy and must be handled with great caution.



CAUTION!

Recycling batteries

Used batteries must be disposed of responsibly in accordance with local recycling regulations.

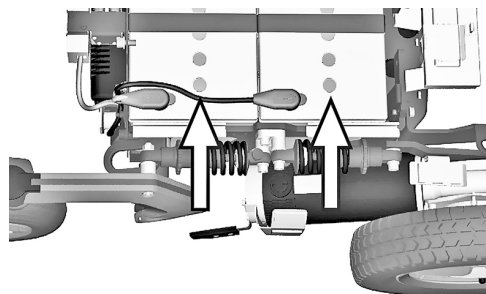


Figure 11. Location of batteries.

5.3.2.5 Main circuit breaker

The wheelchair is equipped with an automatic main circuit breaker that can be reset when it has been tripped.

It also functions as a battery isolator and is controlled, on or off, by using the lever found in a slot in the rear cover of the chassis.



NOTICE

Investigate tripped main circuit breaker

A tripped main circuit breaker often indicates a major electrical fault. The cause of a tripped main circuit breaker must be carefully investigated and determined before resetting the circuit breaker.



NOTICE

Before using the main circuit breaker

Always switch off the power to the control panel before switching off the main power with the main circuit breaker.

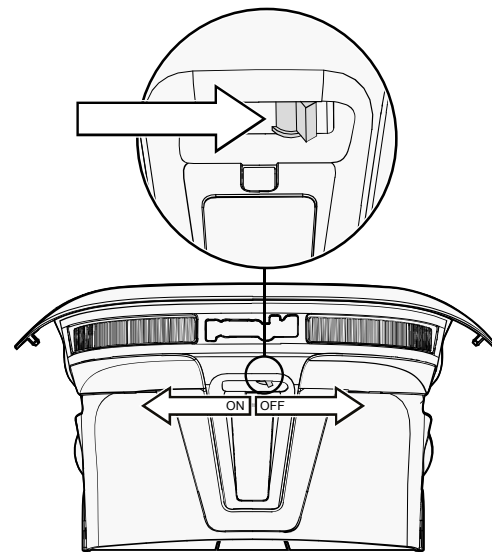


Figure 12. Main circuit breaker location.

5.3.2.6 Charger socket

The charger socket is located at the front of the joystick.



CAUTION!

Switch off main circuit breaker

Always switch off the power supply to the control panel before interrupting the power with the main circuit breaker.

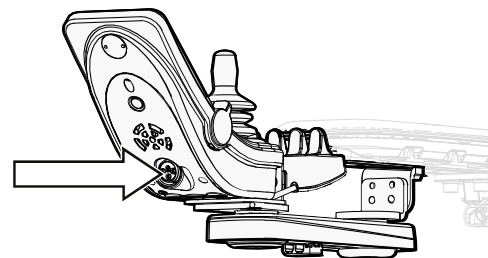


Figure 13. Charger socket.

5.4 Control Panel

5.4.1 Permobil Joystick Module for R-net

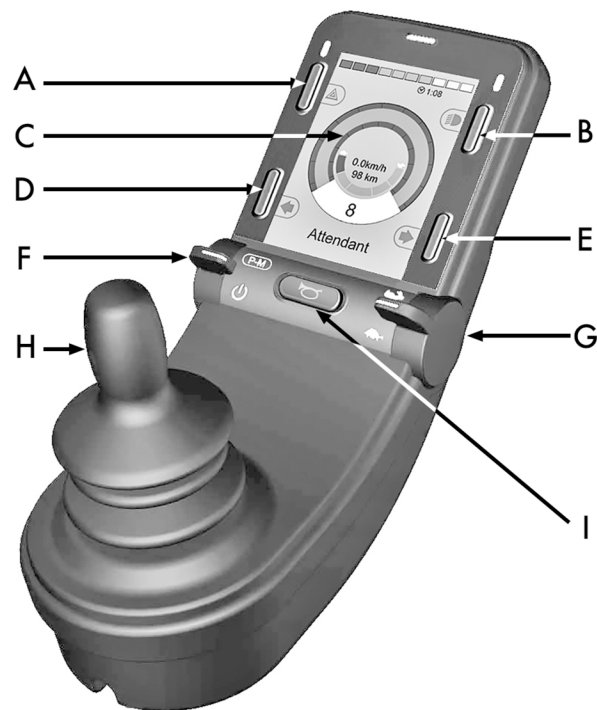


Figure 14. Permobil joystick panel, 2,8" color display.

- A. Hazard flashers on or off.
- B. Headlights on and off.
- C. Screen.
- D. Left turn signals.
- E. Right turn signals.
- F. Main power On or Off; mode or profile selector.
- G. Maximum speed paddle, decrease or increase.
- H. Joystick.
- I. Horn button.

The overview image on the preceding page shows the basic functions of the control panel. All buttons, toggle switches and the joystick may have additional functionality.

The charging socket is located on the front of the panel

Figure 15 *Charger socket.*

Two jack sockets are located on the base of the panel. One is intended for an external On/Offswitch and the other for an external profile switch. Figure 24 *Jack sockets.*

The wheelchair may also be equipped with an extra seat control panel in addition to the control panel.

5.4.1.1 Charger socket

This socket may only be used for charging or locking the wheelchair. Do not connect any type of programming cable to this socket. Do not use the socket as a power supply for any other electrical device. Connection other electrical devices may damage the control system or affect the wheelchair's EMC (electromagnetic compatibility) performance.



NOTICE

Use only the supplied battery charger

The wheelchair's warranty will be voided if any device other than the battery charger supplied with the wheelchair or the lock key is connected via the control panel charger socket.

5.4.1.2 Buttons and paddle switches

On the control panel there is a total of 5 buttons and 2 paddle switches.

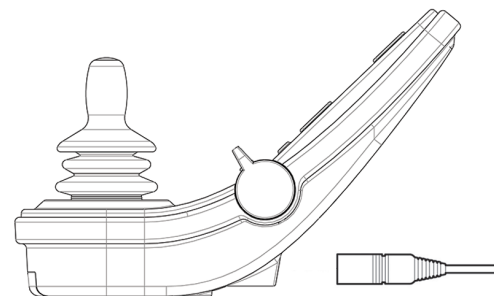


Figure 1 5. Charger socket.

Power, mode and profile paddle

The paddle switch is used to switch the control system on or off.

Push the paddle forward to switch the power on and pull it backwards to switch the power off.

The paddle switch can also be used to scroll through the available profiles and modes. Push the paddle forward to scroll through the available profiles and modes.



Figure 16. Power, mode and profile paddle.

i If your panel is equipped with an early version of the toggle switch as shown in figure 17, follow the description below.

Pull on the paddle switch to switch the power on or off.

The paddle switch can also be used to scroll through the available profiles and modes. Push the paddle forward to scroll through the available profiles and modes.

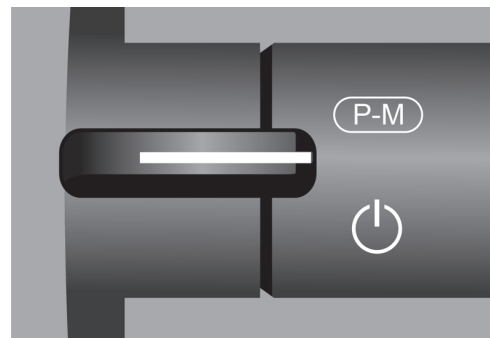


Figure 17. Power, mode and profile paddle switch, early version.

Maximum speed paddle

This paddle decreases or increases the wheelchair's maximum speed. The maximum speed indicator on the display shows the current setting. Push the paddle forward to increase the setting and backwards to decrease the setting.

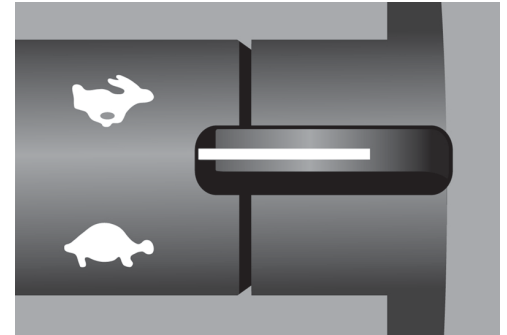


Figure 18. Maximum speed paddle, decrease or increase.

Horn button

The horn will sound while this button is depressed.



Figure 19. Horn button.

Function Buttons

There are a total of four buttons located on the left and right sides of the display. The symbols on the display show the current button function.

Hazard warning button and screen symbol

(i) Only active if the wheelchair is fitted with lights.

This button switches the wheelchair hazards lights on or off. The hazards lights are used when the wheelchair is positioned such that it constitutes an obstruction for others. Push the button to switch the hazard lights on and push it again to switch them off. When activated the screen symbol will flash in sync with the wheelchair hazard lights.

Lights button and screen symbol

(i) Only active if the wheelchair is fitted with lights.

This button switches the wheelchair lights on or off. Push the button to switch the lights on and push it again to switch them off. When on, the lights screen symbol lights up.



Figure 20. Hazard warning button and screen symbol.



Figure 21. Lights button and screen symbol.

Left turn signal button and screen symbol

i Only active if the wheelchair is fitted with lights.

This button switches the wheelchair's left turn signal on or off. Push the button to switch the turn signal on and push it again to switch it off. When activated, the left turn signal screen symbol will flash in sync with the wheelchair's turn signal.



Figure 22. Left turn signal button and screen symbol.

Right turn signal button and screen symbol

i Only active if the wheelchair is fitted with lights.

This button switches the wheelchair's right turn signal on or off. Push the button to switch the turn signal on and push it again to switch it off. When activated, the right turn signal screen symbol will flash in sync with the wheelchair's turn signal.



Figure 23. Right turn signal button and screen symbol.

5.4.1.3 Jack sockets

The external On/Off switch jack (A) allows the user to turn the control system on or off using an external device such as a buddy button.

The external profile switch jack (B) allows the user to select profiles using an external device, such as a buddy button. To change the profile while driving, simply press the button.

5.4.1.4 Display

The status of the control system can be understood by observing the display. The control system is on when the display is backlit.



Figure 24. Jack sockets.

Screen symbols

The R-net drive screen has common components that always appear and components that only appear under certain conditions.

- A.** Speedometer
- B.** Profile name
- C.** Battery indicator
- D.** Clock
- E.** Maximum speed indicator
- F.** Current profile

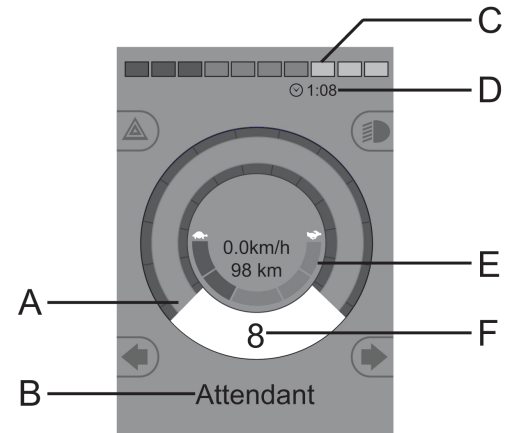


Figure 25. Drive screen when set to profile 8.

Battery indicator

This displays available battery charge and can be used to alert the user of the status of the battery.

- Steady light: everything is in order.
- Flashing slowly: the control system is functioning correctly, but charge the battery as soon as possible.
- Stepping up: the wheelchair batteries are being charged. The wheelchair cannot be driven until the charger is disconnected and the control system is switched off and on again.

Maximum speed indicator

This displays the current maximum speed setting.

The maximum speed setting is adjusted using the speed paddle.



Figure 26. Battery indicator.



Figure 27. Maximum speed indicator.

Current profile

The profile number describes which profile the control system is currently operating in. The profile text is the name or description of the profile the control system is currently operating in.

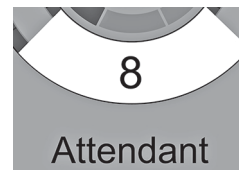


Figure 28. Current profile.

In focus

When the control system contains more than one method of direct control such as a secondary joystick module or a dual attendant module, the module that has control of the wheelchair displays the 'in focus' symbol.



Figure 29. In focus.

Speed limited

If the speed of the wheelchair is being limited; e.g., by a raised seat, then this symbol will be displayed. If the wheelchair is being inhibited from driving, then the symbol will flash.



Figure 30. Speed limited.

Restart

When the control system requires a restart, e.g. after a module reconfiguration, this symbol will flash.



Figure 31. Restart required.

Control system temperature

This symbol means that a safety feature has been triggered. This safety feature reduces the power to the motors, and automatically resets when the control system has cooled down. When this symbol occurs, drive slowly or stop the wheelchair. If the control system temperature continues to increase it can reach a level where the control system must cool down, at which point it will not be possible to drive any further.

Motor temperature

This symbol means that a safety feature has been triggered. This safety feature reduces the power to the motors, and automatically resets after a certain period of time. When the system is reset, the symbol disappears. When this symbol occurs, drive slowly or stop the wheelchair. Permobil recommends that you drive slowly for a short period after the symbol has disappeared, to prevent unnecessary strain on the wheelchair. If the symbol occurs multiple times and the wheelchair is not driven in any of the conditions mentioned in 6.2.4 *Driving restrictions*, Page 193, there might be something wrong with the wheelchair. Contact your service technician.



Figure 32. Control system temperature.



Figure 33. Motor temperature.

Hourglass

This symbol is displayed when the control system is changing states. An example would be entering into programming mode. The symbol is animated to show sand falling.



Figure 34. Hourglass.

Emergency stop

If the control system is programmed for latched drive or actuator operation, then an emergency stop switch is usually connected to the external profile switch jack. If the emergency stop switch is operated or disconnected, this symbol will flash.



Figure 35. Emergency Stop.

Settings Menu

The Settings Menu permits the user to set the clock, display brightness, background color etc.

Press the hazard flasher button for 2 seconds in order to open the menu. Scroll through the menu by moving the joystick back and forth.

A right joystick deflection will enter a submenu with the related function options.

Exit the setting menu by first selecting Exit on the bottom of the menu and then moving the joystick to the right.

The menu items are described in the following sections.

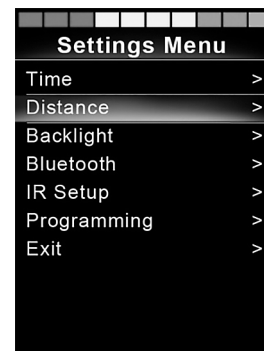


Figure 36. Settings Menu.

Time

The following section describes submenus related to Time.

- Set Time** allows the user to set the current time.
- Display Time** this sets the format of the time display or turns it off. The options are 12hr, 24hr or Off.

Distance

The following section describes submenus related to Distance.

- Total Distance** this value is stored in the power module. It is related to the total distance driven during the time that the current power module has been installed in the chassis.
- Trip Distance** this value is stored in the joystick module; it relates to the total distance driven since the last reset.
- Display Distance** sets whether Total Distance or Trip Distance appears as the odometer display on the joystick module.
- Clear Trip Distance** a right joystick deflection will clear the Trip Distance value.

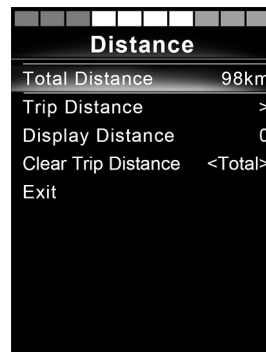


Figure 37. Distance measurement.

Backlight

The following section describes submenus related to Backlight.

Backlight	this sets the backlight on the screen. It can be set between 0% and 100%.
Auto Backlight	the joystick module contains an ambient light sensor to automatically adjust screen brightness. The programmable options are Off or On. Set to On the display adjusts the screen brightness based on the light sensor reading. Set to Off ; screen brightness will not change with changes in light intensity.
Backlight Time	this adjusts the period of time the backlight will remain active once no further instructions are received from an input device, it is adjustable between 0 and 240 seconds.

Bluetooth®

i Applies to control panels equipped with integrated Bluetooth®.

A right deflection of the joystick will enter a submenu to configure the Bluetooth® mode screen. Refer to the Bluetooth® mode chapter for more details, see page 80.

IR Setup

 Applies to control panels equipped with integrated IR.


A right deflection of the joystick will enter a submenu for learning and deleting IR codes. Refer to the IR section for more details; see page 93.

Programming

The following section describes submenus related to Programming.

Sleep	sets the time after which the control system will go to sleep if an input device command is not received.
Sounder Volume	sets the volume of the sounder used to indicate button presses.
Horn Volume	sets the volume of the horn.
Start-up Beep	sets whether not the controller emit a short beep when turned on. Available only in later versions of the control panel.
Momentary Screens	sets whether programmed Momentary Screens are displayed.
Display Speed	sets how the wheelchairs speed is displayed; options are mph, km/h or Off.
Displays	sets the format of the digital drive display; options are odometer, speed or both.

Diagnostics allows the user to read diagnostic information from the control system.

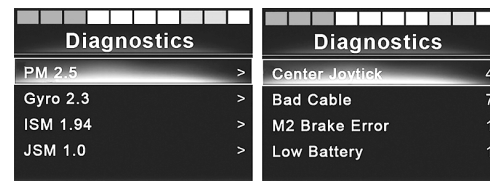

Diagnostics	
PM 2.5	>
Gyro 2.3	>
ISM 1.94	>
JSM 1.0	>

Diagnostics	
Center Joystick	4
Bad Cable	7
M2 Brake Error	1
Low Battery	1

Figure 38. Two samples from the Diagnostic screen.

Timers enables the user to view how many hours the chair has been driven for.

Exit

Exits the Settings Menu.

5.4.1.5 Bluetooth® mode

i Applicable only to R-net control panels, 3,5" or 2,8" color display, with Bluetooth® integrated in the joystick module.

The joystick modules differs in that the 3,5" screen version is equipped with the buttons Mode and Profile, positioned under the screen. Joystick modules with 2,8" screens are more compact and lack these buttons.

General

When a wheelchair is delivered equipped with a control panel containing Bluetooth®, the Bluetooth® functions are pre programmed to a basic setup. This setup is described below.

The basic setup is prepared for pairing of up to four Bluetooth® devices, two PC mice or Android™ devices and two iDevices®, without any additional tools. The setup can be reprogrammed using a programming key and an R-net programmer installed on a computer. With this equipment it is possible to e.g. rename or replace text and icons describing a certain device, activate mouse commands, nudge time, change the cursor speed etc.

Pairing with devices

A device must be set to On from the Settings Menu before it can be paired. Follow the description below.

Settings Menu

Press the hazard flasher button for 2 seconds in order to open the Settings Menu. Select Bluetooth®.

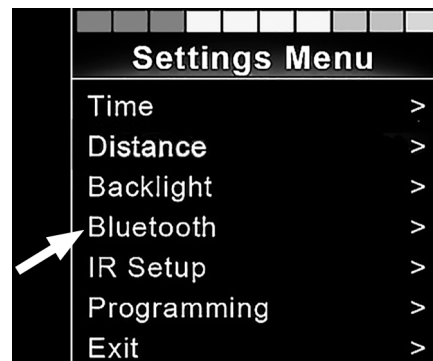


Figure 39. Select Bluetooth® to get access the submenu.

The screen now displays the submenu as illustrated.

Choose a device; set it to On with a right deflection by the joystick.

The R-net system must now be switched off and then on again.

The name of each device can be changed by using a computer with an installed R-net programming tool.

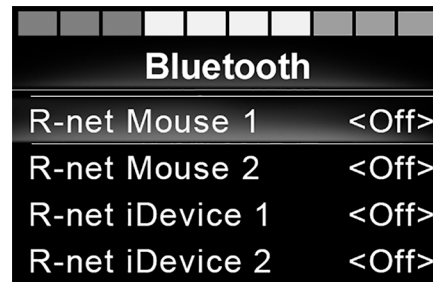


Figure 40. The Bluetooth® submenu.

Pairing setup

The control panel Bluetooth® must then be put into Discovery Mode by following this sequence description:

- Deflect the joystick in the forward direction and hold until there is a beep. This takes approximately 10 seconds; then release.
- Deflect the joystick in the reverse direction and hold until there is a beep. This takes approximately 10 seconds; then release.

The screen will display a flashing Bluetooth® icon above the computer image. This confirms that the control panel Bluetooth® is set in Discovery Mode. It is now discoverable for other Bluetooth® devices.

Pairing with different devices

Depending on the type of device you are pairing with; a PC, an Android™ device or an iDevice®, you will need to follow a specific procedure depending on the device you want to connect to.

Some procedures are described below.



Figure 41. The arrow points to the flashing Bluetooth® icon.

Pairing with a PC

The following process should be carried out on a PC:

- Open the window where you can select a Bluetooth® device to add to the PC. The name and path will differ slightly for different versions of Windows.
- Enter the password 1234 if prompted.
- Click on R-net Mouse X when it appears in the screen and the connection will be finalized. The R-net mouse will now appear in the list of devices on the PC.
- When an connection is established the Bluetooth® icon on the control panel display stops flashing.

Pairing with an Android™ device

The following process should be carried out on the Android™ device

- Select system settings and set Bluetooth® to on.
- Select R-net Mouse X from the list of available devices.
- Enter the password 1234 if prompted.
- R-net Mouse X should now appear as a paired device.
- When an connection is established the Bluetooth® icon on the control panel display stops flashing.

Pairing with iDevice®

The following process should be carried out on the iDevice®.

- Select settings and set Bluetooth® to on.
- Select R-net iDevice® X from the list of available devices.
- R-net iDevice® X should appear as a paired device.
- When an connection is established the Bluetooth® icon on the control panel display stops flashing.

Updating the list of devices

The control panel stores the Bluetooth® IDs of up to four devices.

To replace an entry on the list of devices, one of the existing pairings must be deleted. This process is initiated from the paired device and will vary depending on the type of device.

Once a device is unpaired, a new device can be added.

Operating Bluetooth® devices

This section describes the basic settings of a wheelchair delivered with a control panel with integrated Bluetooth®.

Accessing a Bluetooth® device

The following section describes how to access one of the paired Bluetooth® devices.

To enter Bluetooth® mode

i Joystick module with 2,8" screen is more compact and lacks Mode and Profile buttons.

Press button Mode button (A) a number of times until Bluetooth® mode is entered or press and hold key B for more than 2 seconds to enter Bluetooth® mode, M3.



Figure 42. Two ways to access a Bluetooth® device.

If more than one Bluetooth® device is paired and enabled, a screen will appear where a Bluetooth® device can be selected for connection and use.

If only one Bluetooth® device is enabled it will be shown directly on the screen.

Step between the devices by deflecting the joystick forward or back.

Select the device by right deflection.

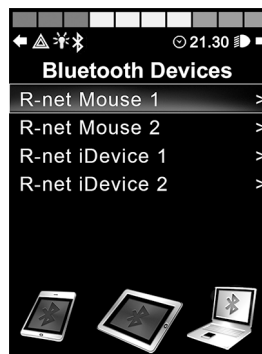


Figure 43. The Bluetooth® Device screen from which paired devices are chosen.

When a Bluetooth® device is selected, the screen will show an icon representing the chosen device type.

If only one Bluetooth® device is enabled, this screen will appear as soon as Bluetooth® mode is activated.

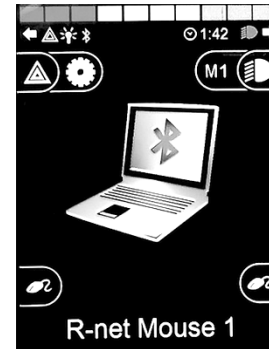
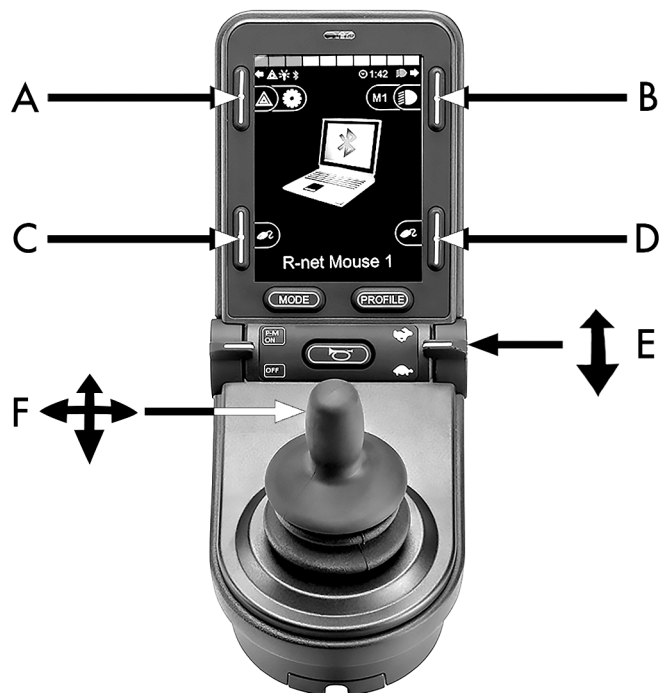


Figure 44. A selected Bluetooth® device.

Operating buttons in Bluetooth® mode

- A. Access settings, page 89.
- B. Return to drive mode, page 89.
- C. Left mouse click, page 90.
- D. Right mouse click, page 90.
- E. Scroll up or down, page 90.
- F. Joystick functions, page 91.

Figure 45. Buttons and other controllers for operating in Bluetooth® mode.

Buttons A to D, paddle switch E and joystick F, as shown in the previous overview illustration, have predefined functions when a Bluetooth® device is selected in Bluetooth® mode. The functions are described below.

A – accessing settings

Press the button for more than 2 seconds. This opens the Setting Menu (Setting Menu); see *Settings Menu*, Page 75.

A short press turns the hazard flashers on or off.



Figure 46. Button to access settings in Bluetooth® mode.

B – return to drive mode

Press the button for more than 2 seconds. This puts the wheelchair into drive mode.

A short press turns head lights on or off.



Figure 47. Button to return to drive mode from Bluetooth® mode.

C – left mouse click

Press button to perform a left mouse click.



Figure 48. Left button for left mouse clicks.

D – right mouse click

Press button to perform a right mouse click.



Figure 49. Right button for right mouse clicks.

E – scroll up or down

Push the paddle switch to scroll up.

Pull the paddle switch to scroll down.



Figure 50. In mode Bluetooth® the right paddle switch is used to scroll up or down.

F – joystick functions

Move cursor in desired direction by deflecting the joystick.

Nudge forward to scroll up.

Nudge back to scroll down.

Left mouse click; left nudge.

Right mouse click; right nudge.

i A nudge is a quick, full deflection; settings can be changed to accommodate your needs using a computer with the programming tool installed.



Figure 51. With the use of Bluetooth® the joystick can take control of the mouse functions.

Notes - Bluetooth® mode settings changed after delivery

Button, toggle switch or joystick	Function

5.4.1.6 IR mode

- i** Applicable only for R-net control panels with integrated IR, 3.5" or 2.8" color display with IR.

General

The infrared control, integrated in the joystick module, makes it possible to replicate commonly used IR devices, such as remote controls for TV, audio, cable, satellite or environmental controls. IR codes can either be learned without any tools or loaded by the PC-based IR configurator software. The programming abilities of the IR configurator entail more than just loading codes.

This manual describes only the learning method.

The wheelchair is prepared for IR mode but there are no IR codes stored in the system at delivery.

IR setup

IR Setup menu can be accessed via the Settings Menu; see *Settings Menu*, Page 75.

On entering the IR Setup menu, the default appliances will appear. When an appliance is selected, its commands are shown.

When a command is followed by a check mark it means that it has a stored IR code. When a command is not checked, its IR code has not yet been stored.

IR codes can be stored or deleted as described in the following sections.



Figure 52. The IR Setup screen.



Figure 53. Commands on the TV MENU screen.

Learning an IR code

The IR receiver is located above the screen on the control panel, marked with an A in the figure.

When learning a code, the remote control device must be held so that its IR transmitter points towards the IR receiver on the control panel, i.e. the little window marked A in the figure.

Learning an IR code procedure

Enter the IR Setup menu.

Select an appliance, e.g. TV – (Device 1) as illustrated. "(Device 1)" shows where the equipment's unique name may appear on the screen.



Figure 54. The IR receiver location.

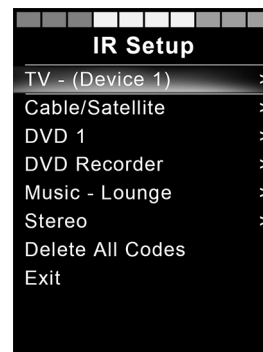


Figure 55. Sample of IR Setup.

The commands for the appliance will appear on the screen as shown in figure. Select a command to learn.

In this example; Channel Up will be selected from the TV MENU.

Select Learn Code, by deflecting the joystick right while the command is highlighted.

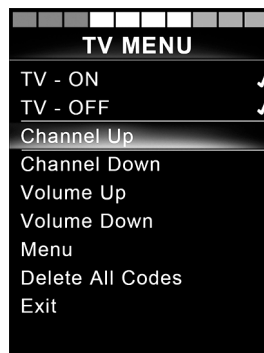


Figure 56. The screen displayed when an appliance has been selected.

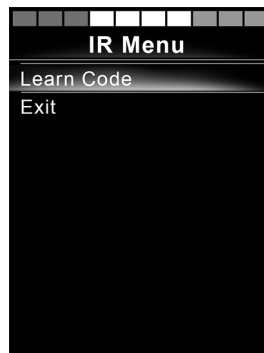


Figure 57. IR Setup menu, Learn Code.

Point the TV remote control towards the control panel's IR receiver and press the channel up button twice.



Figure 58. Point the remote control at the IR receiver. Press the button twice.

If the Learn Code operation has been successfully completed, a green check mark will be displayed on the screen.

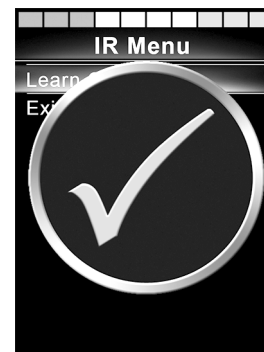


Figure 59. Learning operation successful.

If the Learn Code operation fails, a red cross will be displayed on the screen. Please retry the Learn Code operation.



Figure 60. Learning operation unsuccessful.

After the code has been learned, scroll down to highlight Exit. Deflect the joystick to the left to return to the appliance menu, IR Setup.

- i** The first time an IR code is learned, it is necessary to cycle the power by turning the power off at the control panel and on again. If other IR codes have already been learned, it is not necessary to cycle the power.

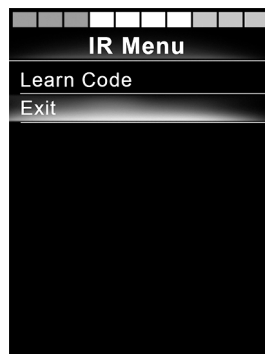


Figure 61 . Scroll down to Exit.

Learning sequenced IR codes

Multiple IR codes can be learned for a single command in the IR Setup menu. This enables multiple IR codes to be transmitted through a single command when in IR mode.

Examples:

1. The on/off function for multiple appliances, the TV and the DVD for example, can be learned by a single entry in the IR Setup menu. The control panel will then transmit the codes for the learned command in one burst. In this case, turning the TV set and the DVD recorder on or off practically simultaneously.
2. Previously, selecting a TV channel required the user to select the individual channel's digits from a list. This could be quite inconvenient when trying to select a TV channel with multiple digits e.g. Channel 143. Now the individual codes for the numbers 1, 4 and 3 can be learned by a single command in the IR Setup menu. When this command is selected in IR Setup menu the IR codes are transmitted practically simultaneously.

Create a sequence

- Select the command to use as the sequence initiator. In this example, TV – ON.
- Select Learn Code, by deflecting the joystick to the right while the command is highlighted.
- Point the TV remote control at the control panel's IR receiver and press the On/Off button twice.
- After each successful learn operation a check momentarily appears on the screen, select Learn Code again.
- Point the DVD remote control at the control panel's IR receiver and press the On/Off button twice.
- After each successfully learned operation, a check momentarily appears on the screen, select Learn Code again.
- Complete the sequence by highlighting Exit and deflecting the joystick to the left.
- Now the TV – ON command will have a check mark and three dots beside it, showing a learned sequence.

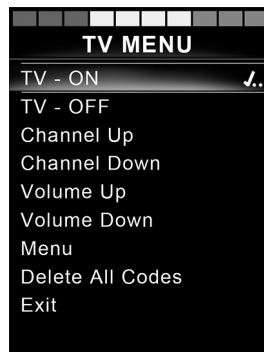


Figure 62. A check mark followed by three dots showing a learned sequence.

Enabling and disabling IR codes

IR codes can be enabled or disabled in the IR Setup menu. If a code is disabled, it will not transmit and will not appear in IR mode.

To disable an IR code, deflect the speed paddle up or down. A disabled IR code appears with an X against the highlighted command.

To enable an IR code, deflect the speed paddles up or down. An enabled code appears with a check mark against the highlighted command.

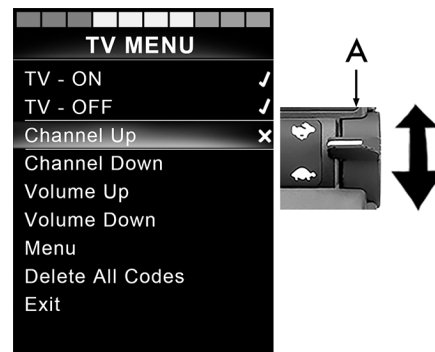


Figure 63. Screen and speed paddle switch marked A.

Deleting IR codes

To delete an IR code for a specific command, highlight the command in the appliance menu and deflect the joystick to the right. Then select the Delete Code option.

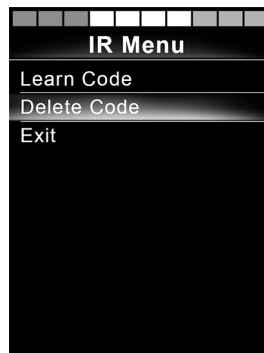


Figure 64. Deleting codes for specific commands.

To delete all IR codes for an appliance, select Delete All Codes within that appliance's submenu.

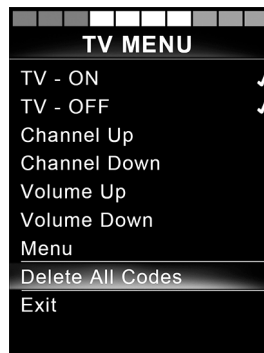


Figure 65. Deleting all codes for a specific appliance.

To delete all IR codes stored in the control panel, select Delete All Codes within the IR Setup menu.

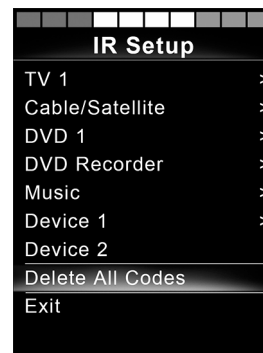


Figure 66. Delete all codes from the IR Setup menu.

5.4.1.7 Locking and unlocking the control system

Locking

- Switch the control system on and move the paddle forward.
- After 1 second the control system will beep. Now release the paddle.
- Deflect the joystick forward until the control system beeps.
- Deflect the joystick rearward until the control system beeps.
- Release the joystick; there will be a long beep.
- The wheelchair is now locked.
- The wheelchair is switched off.



Figure 67. The lock symbol is displayed when the wheelchair is locked.

Unlocking

- If the control system has switched off, move the paddle forward.
- Deflect the joystick forward until the control system beeps.
- Deflect the joystick rearward until the control system beeps.
- Release the joystick; there will be a long beep.
- The wheelchair is now unlocked.

5.4.1.8 Seat functions

Not applicable to all seat models

On some seats, the power functions can be controlled using the control panel joystick. Some models can memorize three seat positions. The seat adjustment mechanism stores each memorized seat position. This makes it easy to retrieve a seat position saved earlier.

Maneuvering the seat

1. Push the mode paddle switch forward one or more times until a seat function icon appears in the control panel display.
2. Move the joystick to the left or right to select a seat function. The icon for the seat function selected appears in the display. The icons shown may vary depending on the seat model and available functions.
3. Move the joystick forward or rearward to activate the function.



NOTICE

The symbol M

If the symbol M appears together with the seat icon, it means the memory function has been activated. Move the joystick to the left or right to choose a seat function instead.

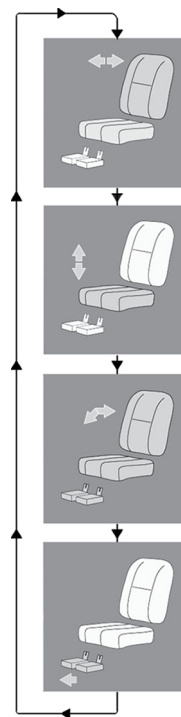


Figure 68. Move the joystick to the left or right to select a function. The icon for the seat function selected appears in the display.

Return to drive mode

Push the mode paddle switch forward one or more times until a standard display image with speed indicator appears in the control panel display.



Figure 69. Standard display image with speed indicator.

Memory

Retrieving position from memory

Some seat control systems can memorize three seat positions. The seat adjustment mechanism stores each memorized seat position. This makes it easy to retrieve a seat position saved earlier.

1. Push the mode paddle switch forward one or more times until a seat function icon appears in the control panel display.
2. Move the joystick to the left or right to select a memorized position (M1, M2 or M3). A seat icon and memory symbol M for the memorized position selected are shown in the control panel display.
3. Move and hold the joystick forwards. The seat adjusts to the position stored earlier. For reasons of safety, the joystick must be held forward until the seat is fully adjusted to the required position. Once the seat has assumed the saved position, it stops moving.



Figure 70. Memory function activated.



NOTICE

Releasing the joystick stops seat movement

Release the joystick at any time to stop seat movement.

Return to drive mode

Push the mode paddle switch forward one or more times until a standard display image with speed indicator appears in the control panel display.

Saving a seat position to memory

1. Set the seat's power functions to the preferred position.
2. If not activated, activate the seat/memory function by pushing the "Mode" paddle switch forward one or more times until a seat icon appears in the control panel display.
3. Move the joystick to the left or right to select a memorized position (M1, M2 or M3). A seat icon and memory symbol M for the memorized position selected are shown in the control panel display.
4. Move the joystick rearward to activate the Save function. An arrow will appear next to the memory symbol M.
5. Save the current position by moving the joystick forwards and holding it in that position until the arrow next to the memory symbol M disappears.

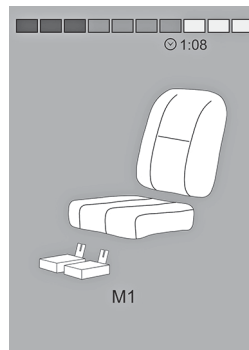


Figure 71. Memory function activated.

Return to drive mode

Push the mode paddle switch forward one or more times until a standard display image with speed indicator appears in the control panel display.

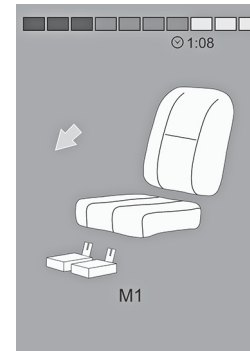


Figure 72. Save function activated.

5.4.2 Joystick panel, R-net LED control panel

5.4.2.1 General

The control panel consists of a joystick and function buttons. At the front of the panel is the charger socket.

The wheelchair may also be equipped with a seat control panel in addition to the control panel.



Figure 73. Control panel.

5.4.2.2 Charger socket

This socket may only be used for charging or locking the wheelchair. Do not connect any type of programming cable to this socket. Do not use the socket as a power supply for any other electrical device. Connecting other electrical devices may damage the control system or affect the wheelchair's EMC (electromagnetic compatibility) performance.



NOTICE

Use only the supplied battery charger

The wheelchair's warranty will be voided if any device other than the battery charger supplied with the wheelchair or the lock key is connected via the control panel charger socket.

5.4.2.3 Function buttons

The control panel has a total of 9 function buttons and a joystick.

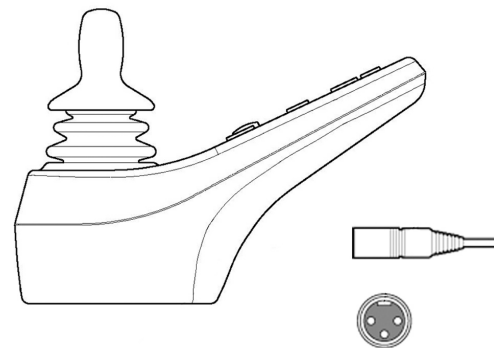


Figure 74. Charger socket.

On/Off button

The On/Off button switches on power to the control system electronics, which in turn supply power to the wheelchair's motors.

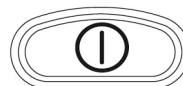


Figure 75. On/Off button.

Horn button

The horn will sound while this button is depressed.



Figure 76. Horn button.

Button and LED for adjusting speed or setting driving profile

This button usually reduces or increases the wheelchair's maximum speed. Alternatively, the button can be used to set the driving profile.

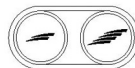


Figure 77. Button and LED for maximum speed or setting the driving profile.

Mode button

These buttons normally reduce or increase the wheelchair's maximum speed. Alternatively, the button can be used to set the driving profile.



Figure 78. Mode button.

Hazard warning button

i Only available if the wheelchair is fitted with lights.

This button switches the wheelchair hazards lights on or off. The hazards lights are used when the wheelchair constitutes an obstruction for others. Push the button to switch the hazard lights on and push it again to switch them off. When activated, the hazard LED will flash in sync with the wheelchair's hazard lights.



Figure 79. Hazard warning button and LED.

Lights button

i Only available if the wheelchair is fitted with lights.

This button switches the wheelchair lights on or off. Push the button to switch the lights on and push it again to switch them off. When on, the lights LED lights up.



Figure 80. Lights button and LED.

Left turn signal button

i Only available if the wheelchair is fitted with lights.

This button switches the wheelchair's left turn signal on or off. Push the button to switch the turn signal on and push it again to switch it off. When activated the left turn signal LED will flash in sync with the wheelchair's turn signal.



Figure 81. Left turn signal button and LED.

Right turn signal button

i Only available if the wheelchair is fitted with lights.

This button switches the wheelchair's right turn signal on or off. Push the button to switch the turn signal on and push it again to switch it off. When activated the right turn signal LED will flash in sync with the wheelchair's turn signal.



Figure 82. Right turn signal button and LED.


5.4.2.4 Battery voltage indicator

The battery voltage indicator does not show exactly how much charge is left in the battery, but it provides a rough idea to help you avoid unnecessary stops due to discharged batteries.

The indicator shows a more precise value about 1 minute after you stop driving and are no longer using any power functions.

**NOTICE****Voltage indicator also used as fault indicator**

The battery voltage indicator also functions as a fault indicator for the wheelchair's electronics. See page 280 for further information.

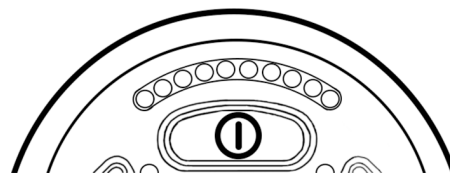**The display shows battery status (from left to right):**

Red, Yellow and Green	Fully charged
Red and Yellow	Half charged
Red	Charge the batteries

Figure 83. Battery voltage indicator.

5.4.2.5 Maximum speed indicator

Speed

Indicates the maximum speed set for the wheelchair.

Driving profile

For special applications, the wheelchair can be programmed with more than one driving profile. In this case, the indicator LEDs will instead display the selected driving profile. There can be up to 5 driving profiles.



NOTICE

Speed indicator also used as a fault indicator

The maximum speed or driving profile indicator also functions as a fault indicator for the wheelchair's electronics. See page 280 for further information.

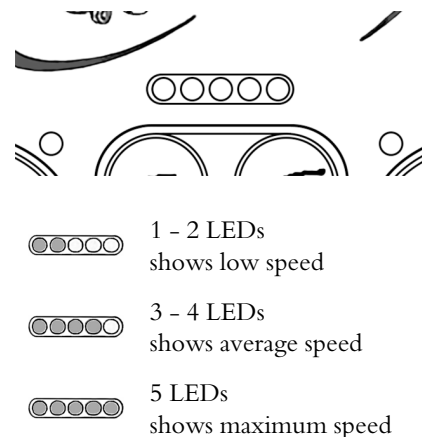


Figure 84. Maximum speed indicator.

5.4.2.6 Seat indicator

On certain seats, the power seat lift, seat angle, backrest angle and leg rest angle functions are controlled by the control panel joystick. In this case, the active seat function is shown on the control panel seat indicator.

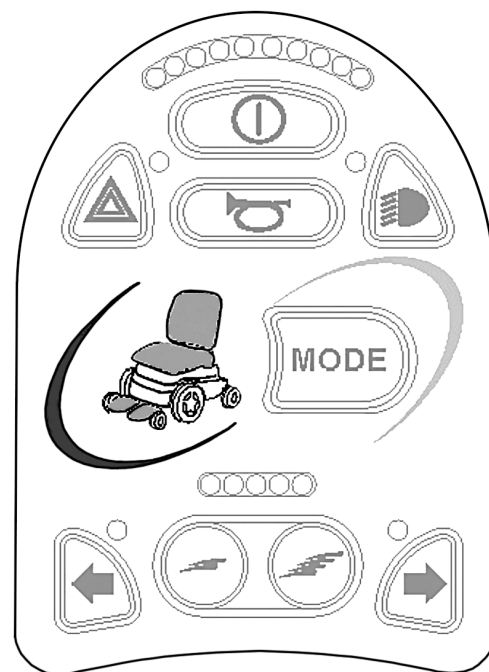


Figure 85. Seat indicator.

5.4.3 ICS seat control panel

5.4.3.1 General

The seat's powered functions may be controlled from the drive system control panel. On seats equipped with the control system ICS, the powered functions may also be controlled from the ICS control panel.

The seat's powered functions are controlled from the control panel. The control system may be in the form of push buttons or toggle switches for those users who find them easier to maneuver. Move the toggle switch forward to operate the front button and back to operate the rear button. The button box functions described here apply to conventional push buttons, but the functions are the same regardless of control system design.

Each switch location can have two functions, and the symbol for the active function is lit. Change functions with the shift function.



NOTICE

Functions vary

The number of available functions will vary depending on how the wheelchair and seat are equipped.



Figure 86. ICS seat control panels, the topmost equipped with push buttons and the lower with paddle switches.

Feedback

The control panel provides feedback regarding the available seat functions, active seat function inhibits, active drive speed limits and active drive inhibits through its LEDs.

The LED above the switch can assume three states:

1. Off

The function icon is extinguished or off. This means the function is not currently active. Toggle selection of the left or right seat functions with the Shift Switch

2. Steady LED

A steady LED provides the user with information concerning drive speed.

- A steady green LED means the chair can be driven at full speed.
- A steady yellow LED signifies that the drive speed is limited due to the position of this actuator.
- A steady red LED means the drive is inhibited due to the position of this actuator.

3. Flashing LED

A flashing LED provides the user with actuator-related information.

- A flashing green LED signifies a special or extended feature.

- A flashing yellow LED means the seat function is inhibited in one direction, due to a safety limit. The switches below the icon will only move the seat in the safe direction.
- A flashing red LED an actuator error has been detected. Depending on the type of error, the switches below may or may not operate the seat function. Note the circumstances when this indication occurs as this information may help your service provider. Contact your service provider for additional assistance, as service may be necessary.

5.4.3.2 Functions



WARNING!

Risk of pinching while using seat functions

There is a risk of pinch-point accidents when using the power seat functions. Make absolutely sure that nothing gets stuck between moving parts. Failure to do so may lead to personal injury.

Seat lift

The seat can be raised by pressing the upper part of the button and lowered by pressing the lower part.



Figure 87. Seat lift.

Backrest angle

The backrest can be reclined by pressing the lower part of the button and brought back by pressing the upper part.

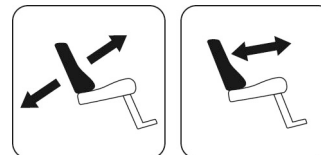


Figure 88. Backrest angle.

Seat tilt

The seat can be tilted backward by pressing the lower part of the button and forward by pressing the upper part.



Figure 89. Seat tilt.

Anterior tilt

The seat can be tilted forward by pressing the upper part of the button and backward by pressing the lower part. When the seat is being tilted forward or backward, the seat will stop when it reaches neutral position. To keep moving forward or backward from neutral position, press the same button again.



Figure 90. Anterior tilt.

Leg rest angle

The leg rest can be tilted forward by pressing the upper part of the button and backwards by pressing the lower part.

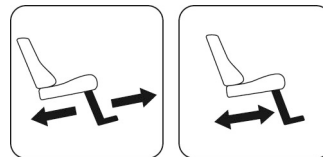


Figure 91. Leg rest angle.

Power adjustable leg length

The footplate can be raised by pressing the upper part of the button and lowered by pressing the lower part.

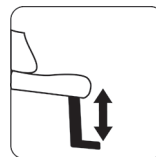


Figure 92. Power adjustable leg length.

Seat-to-floor function

The seat can be lowered to the floor by pressing the upper part of the button, and raised by pressing the lower part.



Figure 93. Seat to floor function.

Standing function

The seat can be raised to standing position by pressing the upper part of the button, and lowered to sitting position by pressing the lower part.

The standing function can be configured to suit each user by pre-programming one of the three possible standing sequences.

The symbol on the left shows the standing sequence which starts by straightening the user's hips and legs before raising the user to a standing position. This sequence minimizes pressure on the user's knees.

The symbol on the right shows the standing sequence which raises the user directly from a sitting position.

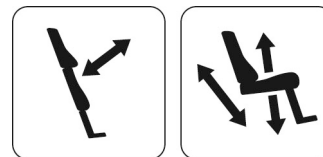


Figure 94. Standing function.

Power footplates

The footplates can be raised by pressing the lower part of the button and lowered by pressing the upper part.

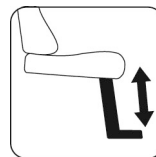


Figure 95. Power footplates.

Power transfer

The footplate can be lowered at the same time as the seat lift is raised by pressing the upper part of the button. Pressing the lower part of the button will raise the footplate and at the same time lower the seat lift. This function can help the user get in and out of the wheelchair.

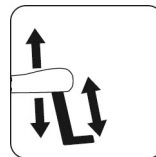


Figure 96. Power transfer footplate.

Shift

Change the control button functions by pressing the Shift button. Change the function back again by pressing the Shift button. The symbol for the active function will light up.

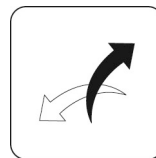


Figure 97. Shift.

Memory function

ICS control panels equipped with memory can store and recall up to three seat positions. A seat function (e.g. power tilt, power recline, power legs, etc.) can only memorize a position if it has an actuator with position feedback installed.

Return to a preset position

1. Enter memory mode by pressing and holding the memory button (8) for two seconds. While in memory mode, the memory LED flashes green.
2. Press and hold the appropriate Recall button (1, 2 or 3) to move the seating system to the preferred preset position. As a safety feature, releasing a Recall button stops actuator movement. When the stored position is reached the actuator movement stops, the LED above the memory position lights up green and the control panel beeps.
3. Return to standard seat function operating mode by pressing the Memory button (8).

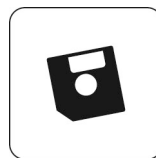


Figure 98. Memory function.



Figure 99. Memory function.

Storing memory

Before storing a seating position in memory, configure the seat system in the desired position.

1. Enter memory mode by pressing and holding the memory button (8) for two seconds. While in the memory mode, the LED flashes green.
2. Press and hold button (4) until the LED immediately to the left of the Memory LED lights up green. Memory is now enabled and ready to memorize a seat position.
3. Press and hold the desired Store button (5, 6 or 7) for three seconds to memorize the current seat position. The LED above the memory position will light up red and the control panel will beep once the memory is successfully stored.
4. Return to standard seat function operating mode by pressing the Memory button (8).



Figure 100. Memory function.

5.4.4 Omni2

Your wheelchair can be equipped with an Omni2. An Omni2 is a device that makes it possible to operate the wheelchair with something other than a standard joystick. For more information, please see the manual on the website of Curtiss-Wright:

<http://support.pgdt.com/omni2-manual.pdf>.

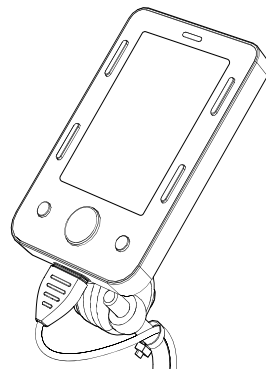


Figure 101. Omni2 display module.

5.5 Accessories

Accessories for Permobil products are subject to continuous development. Contact your nearest Permobil dealer for more information on the accessories available for your product.

5.5.1 ConnectMe

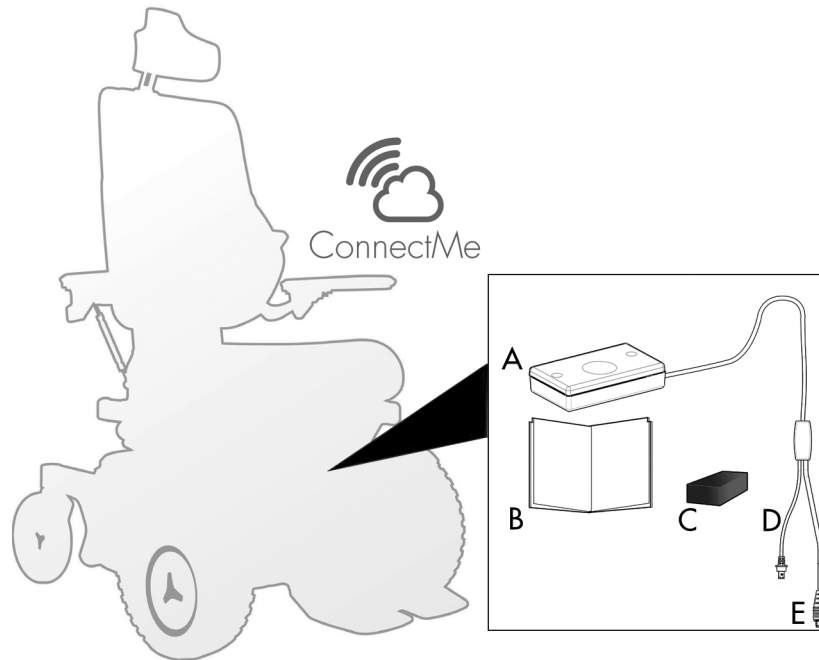


Figure 102. Overview ConnectMe kit.

ConnectMe is a cellular solution that provides communication between the Permobil wheelchair and Permobil cloud service. It is also used in near field communication for transfer of data between the wheelchair and a Bluetooth® device (e.g. smartphone).

5.5.1.1 Federal Communications Commission (FCC) statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

- Consult the dealer or an experienced radio/TV technician for help.

FCC caution: Any changes or modifications not expressly approved by the party responsible for compliance would void the user's authority to operate this equipment.

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

Radiation Exposure statement

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End users must follow the specific operating instructions for satisfying RF exposure compliance.

To maintain compliance with FCC RF exposure compliance requirements, please follow operation instruction as documented in this guide. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter, except the transmitter built-in with this device.

5.5.1.2 Industry Canada statement

ConnectMe complies with the licence-exempt Radio Standards Specifications (RSSs) of Innovation, Science and Economic Development Canada (ISED). ISED is formerly known as Industry Canada. Operation is subject to the following two conditions:

1. This device may not cause interference; and
2. This device must accept any interference, including interference that may cause undesired operation of the device.

5.5.1.3 Prepare for use

In order to make sure that no damage has occurred to ConnectMe while it was being shipped please check the following before use:

- All products ordered are included in the delivery, including manuals and other documentation. If something is missing, please contact your dealer or Permobil for further information.
- No transport or other related damages have occurred to the product. If something has been damaged or appears to be incorrect, please contact your dealer or Permobil for further information as soon as possible.
- If ConnectMe is not behaving normally, power off your wheelchair and contact your dealer or Permobil immediately.

**CAUTION!****Correct installation**

- ConnectMe must be installed by an authorized service technician. If the product is incorrectly installed, it may cause damage to the product, wheelchair, or other property as well as bodily harm to the user.
- ConnectMe must be installed more than 7.87" from the user.
- ConnectMe is solely made for Permobil power wheelchairs. Mounting ConnectMe to equipment other than a Permobil power wheelchair is prohibited. If a user fails to follow this warning, Permobil takes no responsibility for the resulting damage.

**WARNING!****Modifications**

Any unauthorized modifications to ConnectMe or its various systems may increase the risk of personal injury and property damage, including damage to the wheelchair. Do not modify ConnectMe device. Modifications to ConnectMe, may cause damage to the device, the wheelchair, other property and/or bodily damage. Any adjustment to the ConnectMe must be performed by an authorized service technician.

**NOTICE****Storage**

When ConnectMe is not attached to a Permobil wheelchair, store the product in a secure location to prohibit unauthorized access.

**NOTICE****System data**

ConnectMe can record systems data and, if activated, can transmit some data in certain instances, including GPS location. This information helps, for example, to diagnose wheelchair systems and to continuously improve wheelchair safety. Permobil may access the information and share it with others:

- with the consent of the wheelchair owner or his or her personal representative;
- for safety research or wheelchair diagnosis purpose;
- in response to an official request by law enforcement or other government agency;
- for use in dispute resolution involving Permobil, its affiliates or sales / service organization; and
- as otherwise required or permitted by law.

For additional information about how Permobil may use your data, please refer to the privacy notice at <http://www.permobil.com/en/Corporate/Company/Company/Permobil-Privacy-Policy/>

**WARNING!****Airplane mode**

The ConnectMe device contains a radio transmitter. In certain areas radio transmission is not allowed and ConnectMe has to be set to airplane mode, see 5.5.1.5 *Airplane mode*. Examples of these types of areas include, but are not limited to:

- Areas with potentially explosive atmospheres such as fueling areas, below decks on boats, fuel or chemical transfer or storage facilities, areas where the air contains chemicals or particles, such as grain, dust, or metal powders.
- Areas where radio silence is requested such as hospitals, clinics or health care facilities in order to prevent possible interference with sensitive medical equipment.
- Airports or inside airplanes where you are instructed to do so by airport or airline staff.
- Blasting areas or in areas posted with notifications requesting “two-way radios” or “electronic devices” to be turned off to avoid interfering with blasting operations.

The user is responsible for paying attention to where radio transmission is allowed and also where it is not allowed.

5.5.1.4 General use

When mounted on wheelchair, ConnectMe requires no maintenance or operation by the user.

All interaction with ConnectMe is either done through the cloud service web interface or from companion Bluetooth device.

Cleaning

You can clean the ConnectMe device with a soft cloth/sponge if necessary.

Storage

Do not store ConnectMe in rooms susceptible to mist, dampness or water, such as pool areas, laundry rooms, or similar rooms. If you have questions about proper storage of the ConnectMe, contact your supplier or Permobil.

Maintenance and Service

Only perform the minor adjustments and maintenance as specified in the user's manual. All other service, repairs and maintenance must be performed by an authorized service technician.

5.5.1.5 Airplane mode

The ConnectMe device contains a radio transmitter. In certain areas radio transmission is not allowed and ConnectMe has to be set to airplane mode.

Activating airplane mode

1. Turn the power wheelchair ON.
2. Go into seat mode by using the mode button on the joystick or Omni display. If the control panel does not have a mode button use the left paddle switch for mode selection. A switch plugged into the 1/8" monojack port of the joystick or Omni display can also be used for mode selection.
3. If the control panel is an Omni programmed for switched driver controls continue from step 6.

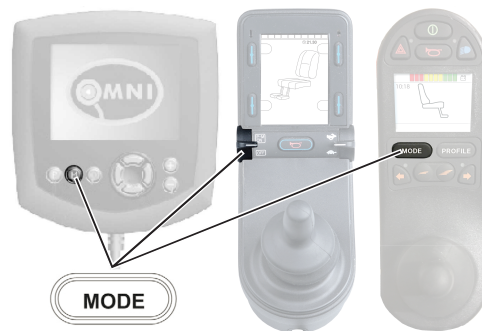


Figure 103. Various ways to access modes.

4. Push and hold the input device left for 15 seconds. A beep will sound.

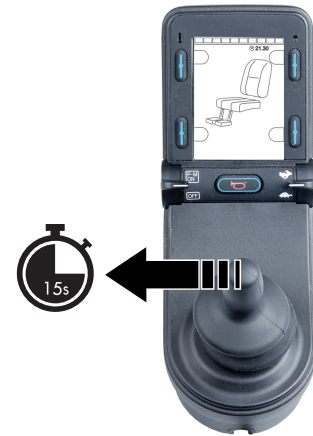


Figure 104. Push and hold left for 15 seconds.

5. The message “Airplane Mode ON” is shown on the display and the ConnectMe’s transmitting modem is off. The message will continue to show on the display as long as the airplane mode is active. However, the wheelchair will work as normal.



Figure 105. Airplane mode is on.

6. Continue from here for an Omni programmed for switched driver controls. Scroll through seat menus until M6 appears and the chair image is shown with yellow arrows.

7. Push and hold the input device forward, or give and hold an equivalent command, for 5 seconds.



Figure 106. Give and hold a forward command for 5 seconds.

8. The yellow arrows will disappear and the message “Airplane Mode ON” is shown on the display and the ConnectMe’s transmitting modem is off. The message will continue to show on the display as long as the airplane mode is active. However, the wheelchair will work as normal.



Figure 107. Airplane mode is on.

Deactivating airplane mode

1. Turn the power wheelchair ON.

2. Go into seat mode by using the mode button on the joystick or Omni display. If the control panel does not have a mode button use the left paddle switch for mode selection. A switch plugged into the 1/8" monojack port of the joystick or Omni display can also be used for mode selection.

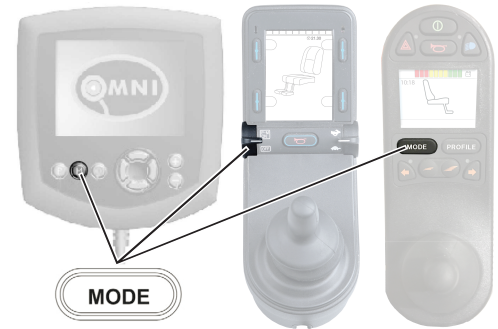


Figure 108. Various ways to access modes.

3. If the control panel is an Omni programmed for switched driver controls continue from step 6.

4. Push and hold the input device left for 15 seconds. A beep will sound.

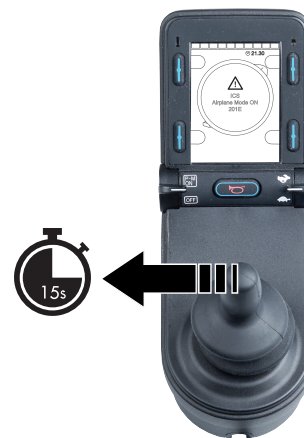


Figure 109. Push and hold left for 15 seconds.

5. The message "Airplane Mode ON" will disappear indicating that the airplane mode is OFF. ConnectMe is now fully operational again.



Figure 1 10. Airplane mode is off.

6. Continue from here for an Omni programmed for switched driver controls. Scroll through seat menus until M6 appears.

7. Push and hold the input device forward, or give and hold an equivalent command, for 5 seconds. Yellow arrows will appear, indicating that the ConnectMe's transmitting modem is back on and that the airplane mode is OFF.



Figure 111. Give and hold a forward command for 5 seconds. Airplane mode is off.

5.5.1.6 Turn off ConnectMe completely

ConnectMe is powered as long as the battery in the wheelchair is connected to its electronics. In order to completely turn off the ConnectMe device the main fuse has to be set in its “Off” state.

5.5.1.7 Market approval

Certifications	
AT&T	Certified for use in AT&T global network
PTCRB:	Certified according to PTCRB in North America
Bluetooth:	Declared Bluetooth smart product

5.5.1.8 Product approval

ConnectMe fulfills the requirements of the following standards:		
	EN 14971 EN 60601-1 + A1 EN 300 328 EN 301 489-1 EN 301 489-7 EN 301 489-17 EN 301 489-24 EN 301 489-52	EN 301 511 EN 301 908-1 EN 303 413 EN 62311 ISO 7176-9 ISO 7176-14 ISO 7176-21 TS 134 124
FCC	FCC 47 CFR Part 15 B FCC 47 CFR Part 15 C (15.247) CFR47 §1.1310 §2.109 KDB 447498 D01	See the backside of the device for the specific FCC IDs
ISED (Industry Canada)	RSS-Gen RSS-247 RSS-102	See the backside of the device for the specific IC IDs

5.5.1.9 Technical specifications

Radio network

GSM:	Quad-band 850 / 900 / 1800 / 1900 MHz
UMTS:	RF Band: B1, B2, B5, B8
Network antenna:	Internal
Connectivity:	Bluetooth 4.1
Connectivity antenna:	Internal

Positioning

GNSS:	A-GPS L1, GLONASS, Galileo
GNSS antenna:	Internal

Dimensions and weight

Dimensions (length x width x height):	3.35" x 1.89" x 0.75" (excluding cable)
Cable length:	3.94"

Dimensions and weight	
External connectors:	6-pin MODU, 4 pin RNET
Weight:	0.13 lb. / 0.32 lb. (including cable)
Rating	
Power operation:	Operational: 24 VDC, I _{max} = 430mA, I _{avg} = 60mA Standby: 24 VDC, Max 0.5 mA
Main fuse:	500mA
Ambient temperature:	Storage: -40°F to +149°F Operating: -13°F to +122°F

5.5.1.10 Serial number label

Label explanations: Numbers in parenthesis refer to QR code GS1 Application Identifiers	
Prod. date (yyymmdd):	Date of manufacturing.
S/N:	Serial number.
Item no:	Permobil internal item number with revision.
HVIN:	Hardware Version Identification Number.
ICC ID:	Integrated Circuit Card identifier.
Supplier ID:	Permobil assigned manufacturer identifier.
This device contains:	This is the FCC/IC identifier of the pre-qualified embedded GSM/UMTS module.
IC:	ISED (Industry Canada) assigned identifier.
FCC ID:	FCC assigned identifier.
Permobil AB:	Manufacturer address.
Permobil Inc.:	Alternative manufacturer address.

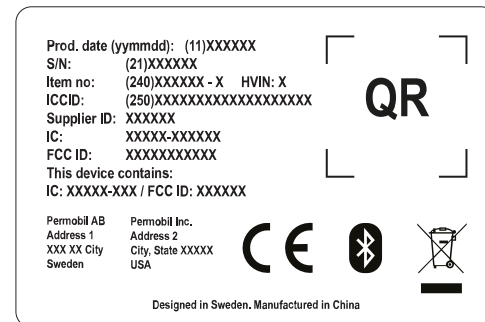


Figure 112. The serial number label of Connect/Me is located on the backside of the enclosure.

5.6 Stickers

Take a good look at all the stickers on the wheelchair and get acquainted with their meaning. The stickers contain important information for safe and proper use.



WARNING!

Accident risk – Always replace missing stickers

Never remove a sticker from the wheelchair. If a sticker becomes difficult to read or falls off, order a replacement sticker from Permobil.

5.6.1 Read the instructions

The sticker indicates that there are instructions that must be read and understood before use or adjustment.

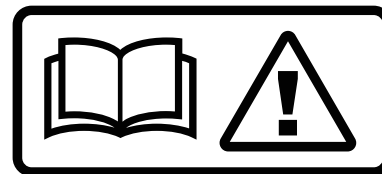


Figure 113. Read the instructions.

5.6.2 Main circuit breaker also battery isolator

The sticker shows the power supply, Off or On.

A description of main circuit breaker function is found on page 264.



Figure 114. Main circuit breaker also battery isolator.

5.6.3 Wheel locks

The sticker shows the wheel lock release lever positions in released and activated states.

A description of wheel lock function is found on page 265.



Figure 115. Wheel locks.

5.6.4 Tie-down point

The sticker indicates where the wheelchair must be secured during transportation. There is a sticker next to each tie-down point.

A description of wheelchair transportation is found on page 207.



Figure 116. Tie-down point.

5.6.5 Support wheels

The sticker indicates that the wheelchair is fitted with support wheels to reduce the risk of tipping when driving over obstacles, etc. They must always be fitted when you drive.



Figure 117. The support wheels must always be fitted when you drive.

5.6.6 Crush hazard

The sticker indicates a crush hazard.



Figure 118. Crush hazard.

5.6.7 Warning

The sticker indicates that special attention is required.

Exercise extreme caution where this warning symbol appears.
Failure to observe warnings may lead to personal injury, including damage to the wheelchair and other property.



Figure 119. Warning, special attention is required.

5.6.8 Battery connections and main circuit breaker

The stickers shows the polarity which differs depending on which battery capacity, 60 Ah or 73 Ah, the product is equipped with.

The positive battery cable (+) is shown in red on the sticker.

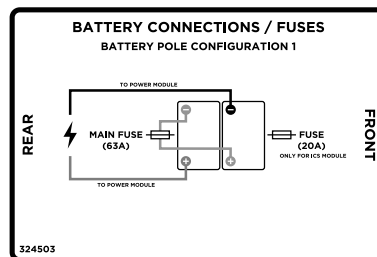


Figure 120. Battery terminals, main circuit breaker and fuse, apply to products equipped with 60 Ah batteries.

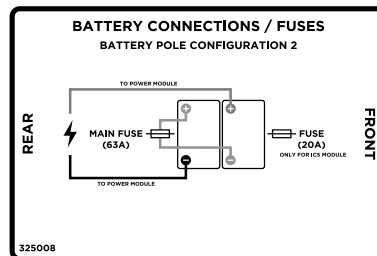


Figure 121. Battery terminals, main circuit breaker and fuse, apply to products equipped with 73 Ah batteries.

6 Preparing for use

6.1	Settings and adjustments	156
6.1.1	Headrest	156
6.1.2	Permobil headrest with link hardware	158
6.1.3	Backrest	162
6.1.4	Chest support	162
6.1.5	Armrest	165
6.1.6	Footplates	171
6.1.7	Knee support	174
6.1.8	Panel holder	178
6.1.9	Positioning belt	183
6.2	Handling and driving	185
6.2.1	General - driving	186
6.2.2	Manual brake release	188
6.2.3	Driving technique	191
6.2.4	Driving restrictions	193
6.2.5	Joystick error	200
6.3	Batteries	201
6.3.1	Charging the batteries	203
6.4	Transporting the wheelchair	207
6.4.1	Vehicle transportation	207
6.4.2	Air transportation	215

6.1 Settings and adjustments

6.1.1 Headrest

 Accessory

This headrest has expanded adjustment options to give the user optimal comfort. The headrest can also be removed and remounted while preserving the same setting.



Figure 122. Headrest overview

6.1.1.1 Removing headrest

1. Undo the handle (C) on the rear of the backrest.
2. Remove the headrest by lifting it straight up.

6.1.1.2 Mounting headrest

Mount in the reverse order

6.1.1.3 Adjusting headrest height and depth

1. Undo the handle (A) on the rear of the backrest.
2. Adjust the height/depth of the headrest as required.
3. Tighten the handle (A).

6.1.1.4 Adjusting headrest angle

1. Undo the handle (B) on the rear of the headrest.
2. Adjust the angle of the headrest as required.
3. Tighten the handle (B).

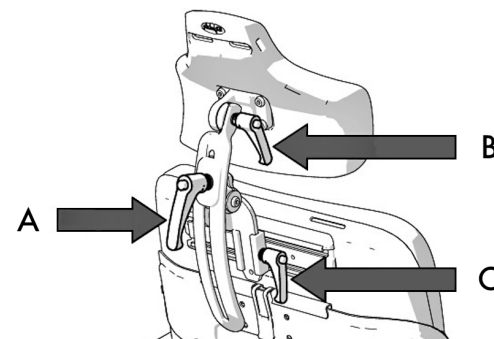


Figure 123. Adjusting headrest



NOTICE

Risk of damaging mechanism

Do not put weight on the headrest while adjusting it.

6.1.2 Permobil headrest with link hardware

Accessory

This headrest has expanded adjustment options to give the user optimal comfort. It is easy to find a correct fit for anyone and the headrest may be removed and installed while preserving the same settings.

6.1.2.1 Removing the headrest

1. Undo the handle (B) at the back of the backrest.
2. Remove the headrest by lifting it straight up (A).

6.1.2.2 Installing the headrest

1. Install in the reverse order.

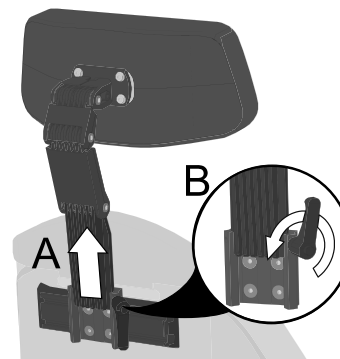


Figure 124. Removal of headrest.

6.1.2.3 Headrest height

1. Undo the handle (B) at the back of the backrest.
2. Remove the headrest by lifting it straight up (A).
3. Adjust the height by changing the placement of the screw on the inside of the bracket. Place the screw in one of the four positions depending on the preferred height. Position 1 provides the lowest setting and position 4 provides the highest setting.

i If position 1 is to be used, all the functions of the chair must be tested using that position to check for interference.

4. Install in the reverse order.

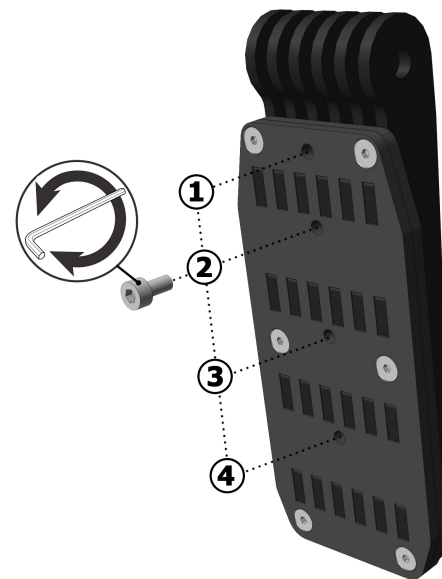


Figure 1.25. Height adjustment screw.

6.1.2.4 Headrest height and depth

1. Undo the screws located at each link.
2. Adjust the height and depth by angling the links as required.
3. Tighten the screws.

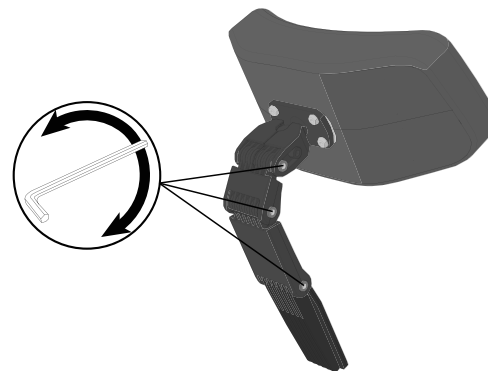


Figure 126. Adjust height and depth.

6.1.2.5 Headrest angle

1. Undo the front screw in the top link (A).
2. Adjust the headrest angle by freely changing the position of the headrest as required (B).
3. Tighten the screw.



NOTICE

Risk of damaging mechanism

Do not put weight on the headrest while adjusting it.

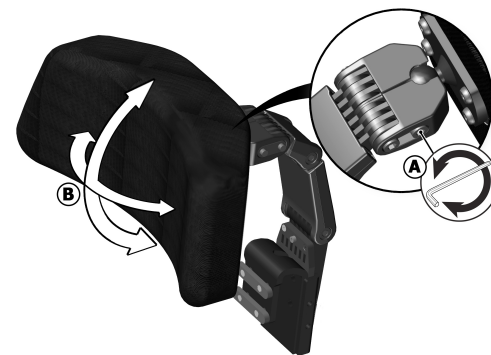


Figure 127. Adjustment of headrest angle.

6.1.3 Backrest

6.1.3.1 Removing backrest

To make transportation of the wheelchair easier, the backrest can be removed in a few simple moves.

1. Remove the backrest cushion. It is fixed in place by means of Velcro on the rear of the cushion.
2. Remove the upper section of the backrest by carefully undoing the clip on the rear of the backrest and pulling the upper section straight up at the same time.

If necessary, the headrest can also be removed; see 6.1.1 *Headrest*, Page 156.

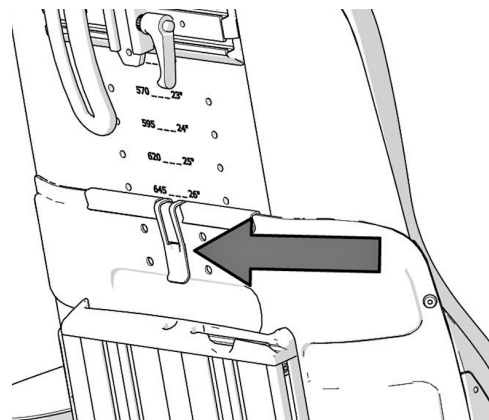


Figure 128. Clip holding the backrest.

6.1.4 Chest support

6.1.4.1 Chest support height

For this task the following tools are necessary:

- 1 Allen key 3 mm.

1. Release the Allen screws (1) on the locking ring on either side of the chest support.
2. Adjust the chest support to a suitable height, but never so high that the fixing is not visible at the lower edge of the sleeve (2).
3. Tighten the Allen screws on the locking ring.

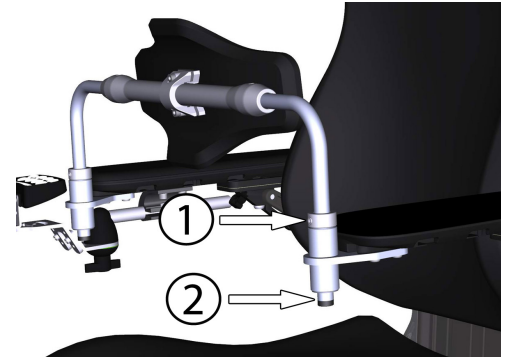


Figure 1 29. Setting the height of the chest support.

6.1.4.2 Chest support depth

For this task the following tools are necessary:

- 1 Torque wrench.
- 1 Allen key 4 mm.

1. Loosen the two screws on the chest support.
2. Position the chest support on a suitable depth.
3. Secure into position by tightening the two screws; torque 17.7 lb.ft.

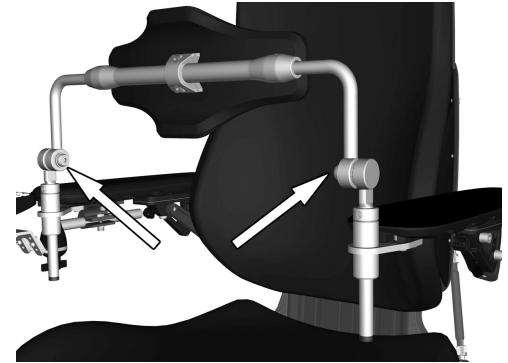


Figure 130. The depth of the chest support is secured with two screws.

6.1.5 Armrest

6.1.5.1 Armrest swivel

i This function is optional

The armrest can be angled inwards or outwards in order to provide optimal comfort for the user. The angle can be changed by pushing the front section of the armrest in or out.



WARNING!

Driving with armrest pointing outwards

Do not pass through narrow passageways with the armrest pointing outwards. This may lead to personal injury and property damage, including damage to the wheelchair.

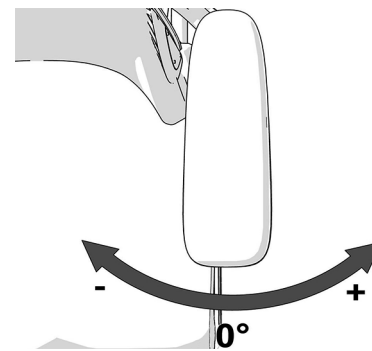


Figure 131. Adjusting the armrest angle.

Setting armrest swivel end position

The end positions are set by moving one or two screws between three different positions underneath the armrest. Fold the armrests up to get easier access to the screws.

- 1. Fold up the armrest.
- 2. Remove the screws underneath the armrest.
- 3. Insert the screws in the preferred positions.
- 4. Fold down the armrest.



Figure 132. Adjusting the armrest swivel end position.

Swivel settings		
A	Factory setting	Locked
B	Factory setting	0° to +15°
C	Custom setting	-15° to +15°
D	Custom setting	0° to +30°

6.1.5.2 Armrest height

The following tools are necessary for this task:

- 1 Allen key, 5 mm.

The height of the armrest is adjustable for optimal comfort. Refer to the scale on the center of the backrest to see the current height of the armrest.

1. Loosen the four screws on the rear of the backrest that secure the height of the armrest.

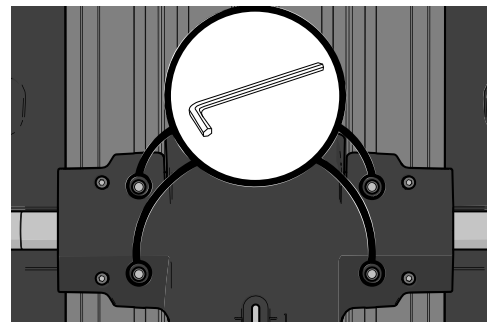


Figure 133. Adjusting the armrest height.

2. Remove the adjustment crank.

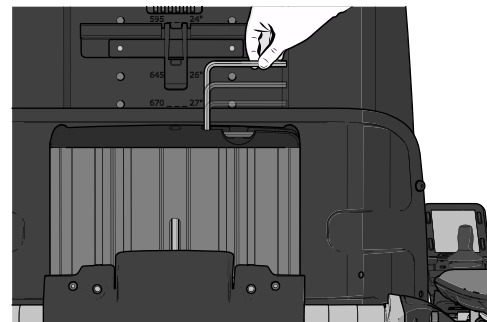


Figure 134. The adjustment crank is located in the backrest profile.

3. Adjust the armrests to the required position using the adjustment crank in the adjustment screw on the rear of the backrest.
4. Secure the height of the armrest by tightening the four screws on the rear of the backrest.



Figure 135. Use the supplied adjustment crank.

6.1.5.3 Armrest angle

The armrests are both individually foldable. The armrest angle can easily be adjusted for optimal comfort.

1. Loosen the two lock nuts on the adjustment bars.
2. Adjust the armrest angle by turning the adjustment bars.
3. Tighten the two lock nuts to secure the adjustment bars in position.



WARNING!

Risk of injury while adjusting armrests

Do not subject the armrests to load while adjusting.

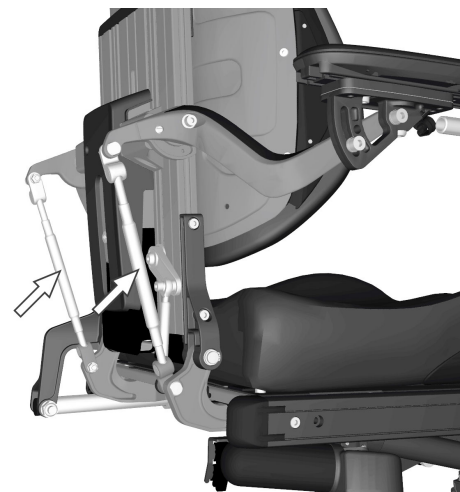


Figure 136. Armrest angle adjustment bars.

6.1.5.4 Armrest height and angle

The following tools are necessary for this task:

- 1 Allen key, 8 mm.

The armrest height and angle is normally adjusted as described previously. However, for special needs, the armrests is adjustable individually for users who want a left and right arm rest at different heights and/or angles. The angle of the armrest is secured using a screw.

1. Loosen the two nuts (D) securing the position of the adjustment bar.
2. Adjust the armrest by turning the adjustment bar (C).
3. Secure into position by tightening the lock nuts (D).
4. Secure the armrest angle by moving the screw from a fixed position (A) to a flexible position (B).
5. Adjust the armrest to the required angle.
6. Secure by tightening the screw (B).

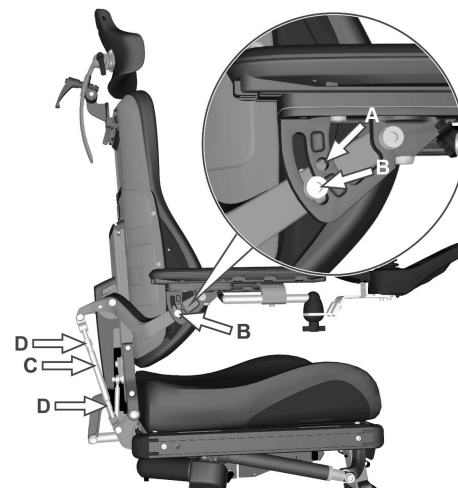


Figure 137. Adjusting the armrest height and angle.

**NOTICE****Armrest flexible position**

This type of adjustment should only be made for special needs. It may have negative effects on the movement of the armrest when raising or lowering the backrest

**WARNING!****Risk of injury while adjusting armrests**

Do not subject the armrests to load while adjusting.

6.1.6 Footplates

**WARNING!****Risk of injury while adjusting footplates**

Do not place any weight or load on the footplates while adjusting the footplates.

6.1.6.1 Footplate height

For this task the following tools are necessary:

- 1 Allen key 5 mm.

The height of the footplates can be adjusted individually and steplessly. They are secured with two screws each.

1. Undo the two screws on the footplate.
2. Adjust the foot plate to the required height and secure by tightening the screws.
3. Check that the footplates are fully secured.



WARNING!

Risk of injury while adjusting footplates

Do not place any weight or load on the footplates while adjusting the footplates.

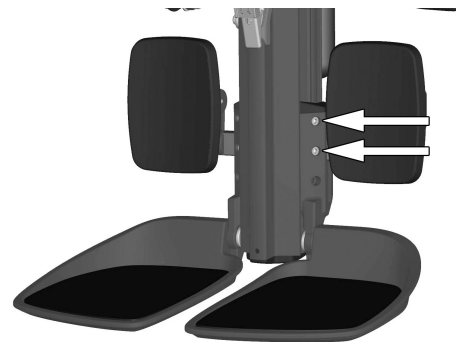


Figure 138. The footplate height are secured with two screws

**WARNING!****Risk of injury - adjust floor to footplate distance**

After adjustment, and with the seat lift in its lowest position, make sure there is sufficient space, at least 1 inch, between the ground and the footplates at all times when maneuvering the leg rest in or out and when using the standing function. Perform this test with the user sitting in the wheelchair loading the footplates.

Using the wheelchair with the footplates set too low might result in personal injury or damage to the wheelchair and its surroundings.

6.1.6.2 Footplate angle

For this task the following tools are necessary:

- 1 Allen key 5 mm.
- 1 Spanner 10 mm.

The angle of the footplates is adjusted using stop screws under each footplate.

1. Tilt up the footplate.
2. Undo the lock nut.
3. Set to the required angle by screwing the screw in or out.
4. Lock the stop screw in the required position using the lock nut.

**WARNING!****Risk of injury while adjusting footplates**

Do not place any weight or load on the footplates while adjusting the footplates.

6.1.7 Knee support

For this task the following tools are necessary:

- 1 Allen key 4 mm.
- 1 Allen key 5 mm.

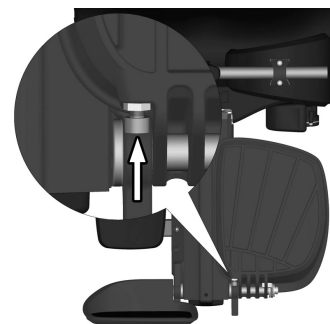


Figure 139. Adjusting the footplate angle

**WARNING!****Leg rest actuator – do not solely use**

You can not solely operate the leg rest actuator when the knee support is fitted. If the actuator is solely operated while the knee support is fitted it may lead to serious injury.

**WARNING!****Adjustment by authorized technician**

This product has to be adjusted by an authorized service technician. If the product is wrongfully adjusted it can cause damage to the user and/or the product.

6.1.7.1 Knee support depth

The depth of the knee support can be adjusted to give the user optimal comfort.

Rotate the knob clockwise, or counter clockwise, to adjust the knee support to a suitable depth.

It should be approximately 1 inch of space between the knee support and the leg when adjusted correctly in a seated position.



Figure 140. Adjusting the knee support depth.

6.1.7.2 Knee support width

The width of the knee supports can be adjusted to give the user optimal comfort.

Loosen the two screws and adjust the knee support pad to a suitable position. Retighten the screws to fix into position.

You can customize how embracing the knee pads are by bending it with your hands.

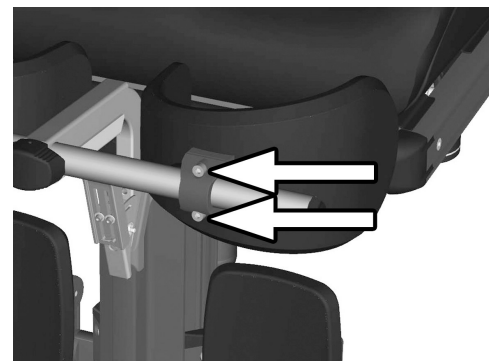


Figure 141. Adjusting the knee support width.

6.1.7.3 Knee support height



WARNING!

Risk of injury - check tube lock

Check that the tube is locked in position by pulling on the support frame upwards. Failure to lock the tube correctly may lead to personal injury.

The height of the knee supports can be adjusted to give the user optimal comfort.

Remove the screw and position the quick lock to a suitable position. Refit the screw to fix into position.

6.1.8 Panel holder

(i) The control panel holder can be mounted on the left or right armrest.

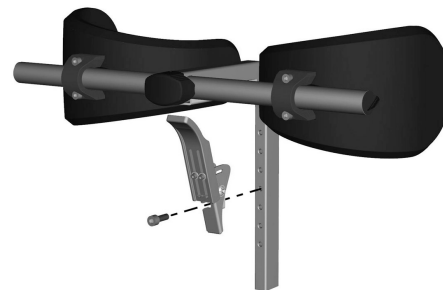


Figure 142. Adjusting the knee support height.

6.1.8.1 Rotational panel holder

The location of the control panel is adjustable lengthwise for the optimal driving position. It is also possible to adjust the angle of the panel sideways to facilitate getting in and out of the wheelchair.

Length adjustment

1. Undo the screw (A) on the panel joint and adjust the panel to the required position.
2. Tighten the screw.

Angle adjustment with friction joint

Using the knob (B) on the friction joint, it is possible to adjust how easily the panel can be pushed out to the side.

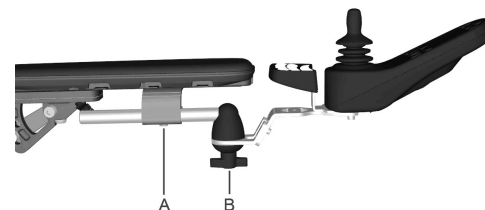


Figure 1 43. Rotational panel holder overview

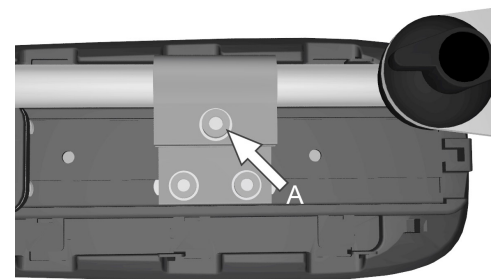


Figure 1 44. Screw for adjusting the panel holder length

Control panel sliding angle adjustment

1. Remove one of the screws. Choose the side that is desired to be sliding.

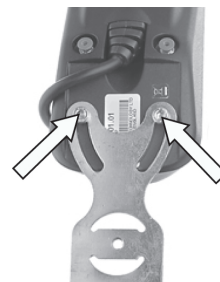


Figure 145. Only remove one of the screws.

2. Angle the panel.
3. Refit the screw. Tighten the screw to the preferred friction.

6.1.8.2 Parallel panel holder

- Allen key, 4 mm.
- Allen key, 5 mm.



Figure 146. The slide tracks.

Length adjustment

1. Undo the screw(s) on the underside enough to slide the panel holder.
2. Adjust the panel to the preferred position. Leave at least a gap of 0.4 inches between the armrest and the panel.
3. Tighten the screw. Tightening torque 7.2 lb.ft.

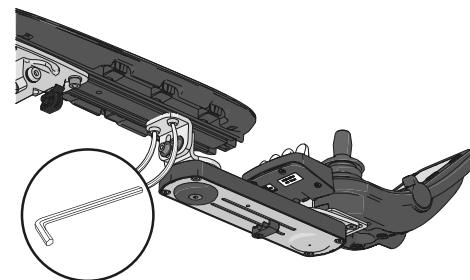


Figure 147. The position of the screws on the parallel panel holder.

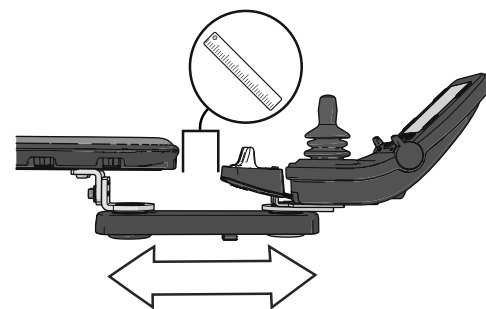


Figure 148. Adjusting the position on the parallel panel holder.

Adjusting the friction joint

1. Undo the screw to make the friction joint more loose.

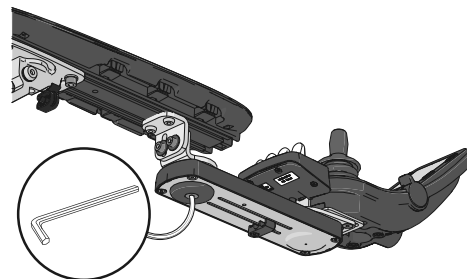


Figure 149. The screw's position on the parallel panel holder.

2. Slide the panel to the preferred position.

3. Tighten the screw to keep it in position.

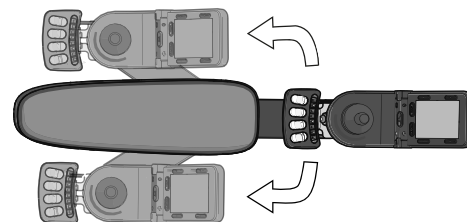


Figure 150. Slide the panel and the panel holder to preferred side of the armrest.

6.1.9 Positioning belt

Accessory

There is an accessory rail on each side of the seat frame intended for attaching a positioning belt. etc. The positioning belt should be fitted in the rail's upper groove.

1. Screw the belt in place, with the snap lock on the side which best suits the user and the other part with the buckle on the opposite side.
2. Check that the belt buckle locks properly in the snap lock.



Figure 151. Attaching the positioning belt



Figure 152. Belt buckle locked properly in the snap lock.

**WARNING!****Use installed positioning aids**

The user must use and securely fasten the positioning belt and any other positioning aids on the wheelchair. Failure to properly use positioning belts and other positioning aids may cause bodily injury. If there is any sign of damage or wear on any positioning belt or aids, immediately contact Permobil for a replacement.

**WARNING!****Positioning belts and aids are not vehicle seatbelts**

The wheelchair positioning belt and aids are designed to properly position the user in the wheelchair for safely operating the wheelchair. The wheelchair positioning belts and aids do not provide protection in a traffic accident and do not replace a vehicle mounted seatbelt.

6.2 Handling and driving

6.2.1	General - driving	186
6.2.2	Manual brake release.....	188
6.2.3	Driving technique.....	191
6.2.4	Driving restrictions	193
6.2.5	Joystick error	200

The wheelchair is designed for use both indoors and out. When driving indoors, take care in e.g. narrow passages, when passing through doors and entrances and when using lifts, ramps, etc.

Also bear in mind the crush hazard when using the power seat lift and seat angle functions, especially if the wheelchair has been driven under tables, benches, etc. When driving outdoors, remember to drive very slowly down steep slopes and to take great care when driving on uneven surfaces, up slopes, on side slopes and over obstacles. Always observe a good safety distance when driving near edges and drops.

We recommend users make repeated test drives in environments where they feel safe to familiarize themselves with how the wheelchair and its accessories behave in different situations before starting to use the wheelchair on public roads and in other public spaces.

6.2.1 General - driving

Check that the control panel is correctly fitted and the joystick is in the neutral position. Make sure you have good support, for example the wheelchair's armrest, for the part you use to handle the joystick. Do not use the joystick alone as a support. Fast turns and driving on uneven surfaces can interfere with your ability to handle the wheelchair safely.

1. Switch on the power.
2. Select a suitable driving profile with the Profile button (if the system is programmed for more than one driving profile).
3. Move the joystick carefully forward to drive forward or backward to back up.
4. Adjust the speed setting with the higher and lower speed buttons. The wheelchair's electronics allow very slow driving over obstacles. Drive up to the obstacle, then carefully drive over it.

**WARNING!****Risk of injury - place your feet correctly**

Always make sure your feet are correctly and securely positioned on the footplates before you operate the wheelchair. Use foot straps if necessary. Failure to do so increases the risk of personal injury.

6.2.2 Manual brake release

The wheelchair is fitted with two manual brake releases acting on the magnetic wheel locks to allow manual movement of the wheelchair.



WARNING!

Wheel locks

Do not release or engage the wheel locks unless power to the wheelchair is off.

To prevent the wheelchair rolling away, make sure it is on a level surface before releasing the wheel locks.

Be aware that the wheelchair has no brakes when the wheel locks are in free-wheel position.

Make sure that the person pushing the wheelchair has full control when the wheel locks are released.

Always engage the wheel locks after the wheelchair has been pushed manually.

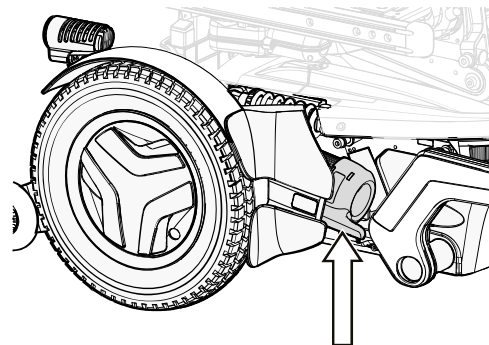


Figure 153. Release lever location.

6.2.2.1 Releasing wheel locks

1. Switch off the wheelchair using the start button on the control panel.
2. Move the lever downwards to release the brakes. The wheelchair can now be moved manually.



WARNING!

Malfunctioning wheel locks

When the wheel locks are released, the wheelchair cannot be driven. If it can be driven, contact an authorized Permobil service center as soon as possible.

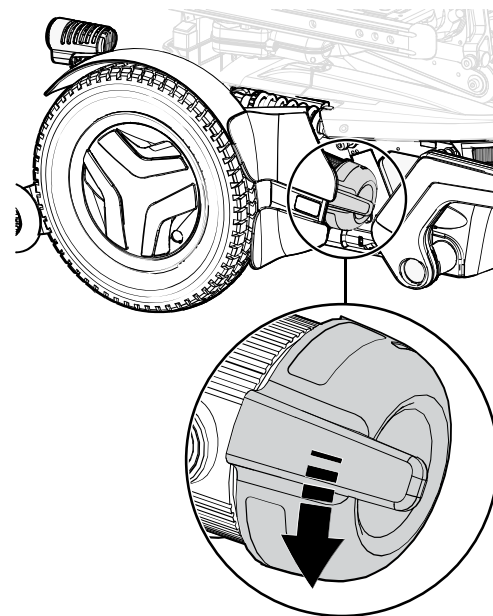


Figure 1 54. Releasing wheel lock.

6.2.2.2 Engaging wheel locks

1. Switch off the wheelchair using the start button on the control panel.
2. Move the lever upwards to its uppermost position to engage the wheel lock.

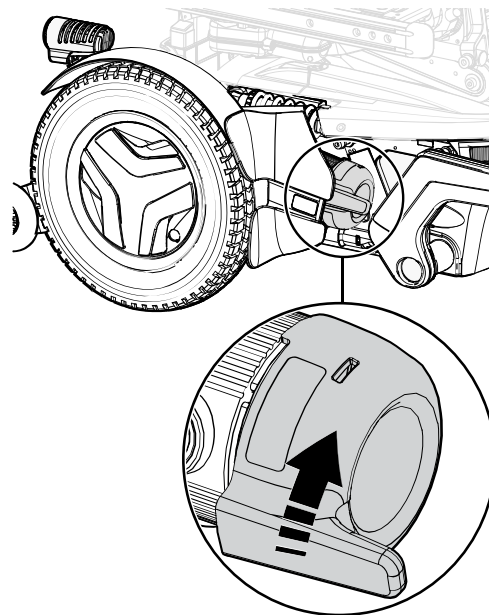


Figure 155. Engaging wheel locks.

6.2.3 Driving technique

The control panel electronics interpret joystick movements and move the wheelchair accordingly. No complex user techniques are required for normal driving, which is an advantage if the user is inexperienced. A good way to get started is simply to move the joystick in the direction you want to go. The wheelchair will then move in the direction in which the joystick is pointing.

However, always remember to drive as gently as possible and to avoid sudden braking and evasive maneuvers.

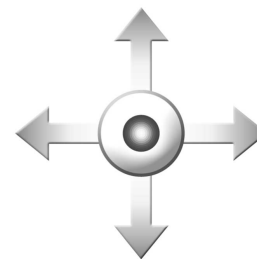


Figure 156. Joystick maneuvering.



CAUTION!

First test drive

Do not take the first test drive alone. The test drive is intended to establish how you and the wheelchair work together and you may need assistance.

Before driving, check that the brake release lever is in the drive position.

**CAUTION!****Driving - surface conditions**

Never drive at full speed in narrow or confined spaces or on sloping or inclined surfaces. Use extra caution when driving the wheelchair on surfaces that have ridges that could cause the chair to steer in a different direction.

**WARNING!****Release the joystick at the first sign of unexpected behavior!**

Do not use the joystick as a support. Fast turns and driving on uneven surfaces can interfere with your ability to handle the wheelchair safely.

If the wheelchair moves in a way that you do not want, **release the joystick!** This always makes the wheelchair stop whatever it is doing.

6.2.4 Driving restrictions



WARNING!

Risk of tipping over

Do not allow the leg rest to hit the ground when climbing obstacles, driving on uneven surfaces or slopes or when a slope levels out. Make sure there is sufficient ground clearance to avoid the risk of tipping over.

The wheelchair consumes much power when you drive up a hill, climb over obstacles, or drive on rough terrain (for example sand). Permobil recommends that you, as far as possible, avoid to drive for a long period of time in these types of conditions.

6.2.4.1 Driving restrictions related to different surfaces

Do not drive at high speed, avoid sudden stops and starts; avoid sudden evasive maneuvers or sharp turns and use extreme caution in any of the following situations:

- when near edges and drops or on elevated surfaces,
- when on soft or uneven surfaces; such as grass, gravel, sand, ice, snow,
- when driving from an area of high traction (e.g. sidewalk) to an area of low traction (e.g. grass),
- when turning the wheelchair,
- when in narrow or confined spaces, and
- when driving on surfaces that have ridges that could cause the wheelchair to move in a different direction.

These restrictions also apply to subsequent sections.

6.2.4.2 Driving over obstacles

You may drive over obstacles up to 3 inches. If a car lock is installed on the wheelchair, it affects the ground clearance.

Use extreme caution when negotiating obstacles of this type. Always approach the obstacle at a right angle. If you drive over higher obstacles, there is a greater risk of tipping and of damage to the wheelchair.



Figure 1 57. Driving over obstacles.



WARNING!

Driving - high obstacles

Do not drive the wheelchair over obstacles higher than 3 inches. Always exercise great caution when driving over obstacles.

Because a raised seat lift, seat angle and/or back angle changes the center of gravity and increases the risk of tipping, only use these seat functions on flat surfaces and always drive with great caution and at low speed.

6.2.4.3 Driving on side slopes

Always exercise great caution when driving over side slopes.

Avoid sudden evasive maneuvers and never drive so fast that you are unable to control the wheelchair safely and without risk.



WARNING!

Risk of tipping on uneven surfaces

Take great care when driving on side slopes with an uneven surface (e.g. grass, gravel, sand and snow).



WARNING!

Risk of tipping on steep slopes

Do not drive the wheelchair on side slopes steeper than 10°. There is a risk of tipping.

Because a raised seat lift, seat angle and/or back angle changes the center of gravity and increases the risk of tipping, only use these seat functions on flat surfaces and always drive with great caution and at low speed.



Figure 158. Driving on side slopes.

6.2.4.4 Driving downhill

Always drive downhill at low speed and with great caution.

Note that the distance required to stop will increase when driving downhill.

Avoid braking suddenly and sudden evasive maneuvers and never drive so fast that you are unable to control the wheelchair safely and without risk.



Figure 159. Driving downhill.



WARNING!

Risk of tipping on uneven surfaces

Take great care when driving on side slopes with an uneven surface (e.g. grass, gravel, sand and snow).

**WARNING!****Risk of tipping when driving downhill**

Do not drive downhill on gradients greater than 10°.

Because a raised seat lift, seat angle and/or back angle changes the center of gravity and increases the risk of tipping, only use these seat functions on flat surfaces and always drive with great caution and at low speed.

**WARNING!****Increased stopping distances**

The distance required to stop the wheelchair will increase when driving downhill.

6.2.4.5 Driving uphill

Always drive uphill with great care.

Avoid sudden evasive maneuvers and never drive so fast that you are unable to control the wheelchair safely and without risk.



Figure 160. Driving uphill.



WARNING!

Risk of tipping on uneven surfaces

Take great care when driving on side slopes with an uneven surface (e.g. grass, gravel, sand and snow).



WARNING!

Risk of tipping when driving uphill

Do not drive uphill on gradients greater than 10°.

Because a raised seat lift, seat angle and/or back angle changes the center of gravity and increases the risk of tipping, only use these seat functions on flat surfaces and always drive with great caution and at low speed.

6.2.5 Joystick error

6.2.5.1 Joystick error at startup

Do not move the joystick before, during or immediately after the control system is switched on. If the joystick is moved from the central position, a joystick error may be generated.


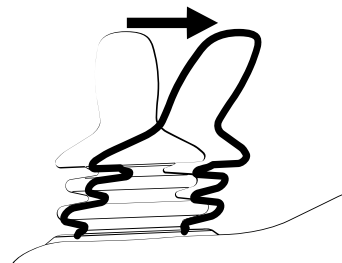


Figure 161. Joystick moved from the central position.

6.2.5.2 Joystick error LCD panel

While the control system checks if an error has occurred, the screen image for a shifted joystick will be displayed for 5 seconds.

If the control system detects an error in the wheelchair electronics, a diagnostics screen will appear. To enable the wheelchair to be driven again, make sure the joystick is in the central position. Then switch the wheelchair off and on again. If this does not work and the diagnostic screen still appears, contact your service provider.

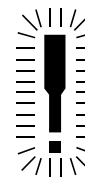

PM
Low Battery

2C00

Figure 162. The LCD diagnostics screen.

6.2.5.3 Joystick error LED panel

While the control system checks if an error has occurred, the LEDs on the battery voltage indicator will wander backward and forward.

If the control system detects an error in the wheelchair electronics, the battery voltage indicator LEDs will flash rapidly. To enable the wheelchair to be driven again, make sure the joystick is in the central position. Then switch the wheelchair off and on again. If this does not work and the LEDs still flash rapidly, contact your service provider.

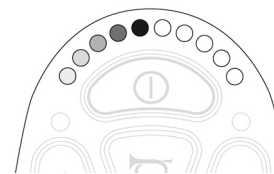


Figure 1 63. The LEDs on the battery voltage indicator.

6.3 Batteries

How frequently you need to charge the wheelchair's batteries will depend on a number of factors including how you use the wheelchair, the temperature and the age of the batteries. All batteries gradually lose capacity as they age.

The most important factor for the life of the batteries is how much power is taken out of them before they are charged and how often they are charged or discharged.

**NOTICE****Discharged batteries**

Should the batteries be drained completely, charge them again as soon as possible since completely drained batteries may reduce battery service life.

To achieve the best service life, never let batteries discharge completely. Always charge the batteries immediately after they have been discharged.

If the battery voltage indicator shows that the batteries lose power faster than normal, the batteries could be worn out and need to be replaced.

6.3.1 Charging the batteries

If the wheelchair is switched on during charging process, the battery voltage indicator bars on the wheelchair control panel will climb to show that the battery is being charged.

If the power on the wheelchair is turned On during the charging process, it must be switched off again and the charger disconnected before the wheelchair can be driven. Use the On/Off button on the control panel.



NOTICE

Battery charger - see supplied manual

For a description and battery charger instructions, refer to the manual supplied.

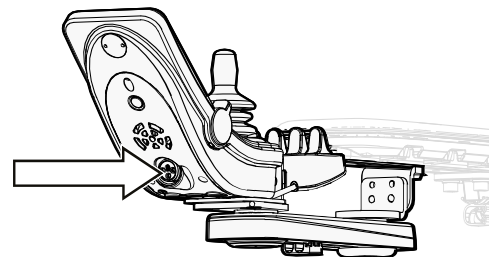


Figure 164. Charger socket.

**WARNING!****Risk of fire or explosion - charging conditions**

Use only the charger supplied with your wheelchair or a charger recommended by Permobil. The use of other chargers may damage the batteries, the wheelchair electronics or the charger itself. It may also result in parts becoming overheated, which may entail a greater risk of fire.

Only use chargers with a max 10 A charging current (average value). The effective value of the charging current must not exceed 12 A. The batteries must be charged in a well-ventilated room, not in a closet. The batteries must not be charged in a bathroom or wet room.

**WARNING!****Risk of fire or explosion - do not short-circuit**

Exercise caution when using metallic objects during work with batteries. A short-circuit can easily cause an explosion. Always use safety gloves and goggles.

**WARNING!****Risk for overheating, sparks or short-circuit**

To interrupt the charging process, turn off the power supply at the switch on the charger before removing the charging jack from the wheelchair's charger socket. This is to avoid sparking and unnecessarily high wear on the charging contact.

The charger may get hot and must not be covered. The charger must be placed so that it has free space on all sides.

The charging jack must be replaced if it is damaged or gets hot during charging. Both the jack on the charger cable and the wheelchair's charger socket must be replaced if one part is damaged or worn. The contacts must be replaced by qualified personnel.

**NOTICE****Procedures when charging the batteries**

The charging cable may not be extended.

Carefully read the instructions supplied with the charger before starting to charge the wheelchair.

The main circuit breaker must be set to On to charge the batteries.

Make sure the charger connector is fully inserted in the wheelchair's charger socket.

The wheelchair cannot be driven when the charger is connected. If it is possible to drive the wheelchair during charging, contact Permobil or your dealer.

6.4 Transporting the wheelchair

6.4.1 Vehicle transportation

The wheelchair may only be transported in vehicles approved for this purpose.

Check that the wheelchair is properly secured and that the wheel locks are engaged. When transporting it in a vehicle, the wheelchair must be secured in position by attaching straps to the tie-down eyes at the front and rear, marked with yellow stickers.

Secure the wheelchair according to the instructions from the manufacturer of the approved vehicle restraint system. Always make sure that the tie-down points in the transporting vehicle are well-anchored.

- i** To make transportation of the wheelchair easier, the backrest can be removed or folded down in a few simple moves; see 6.1.3.1 *Removing backrest*, Page 162
If necessary, the headrest also can be removed; see 6.1.1 *Headrest*, Page 156



Figure 1 65. Sticker showing the location of the tie-down eyes.



Figure 1 66. The wheelchair has four tie-down points. The wheelchair and tie-down position in this picture is generic.

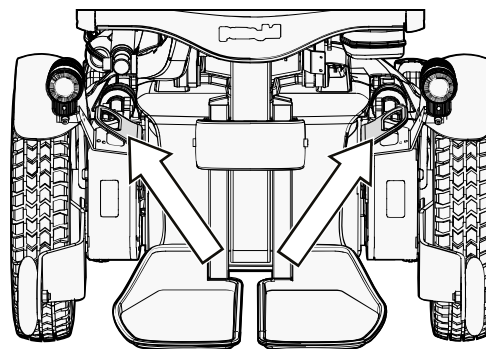


Figure 167. Front tie-down eyes.

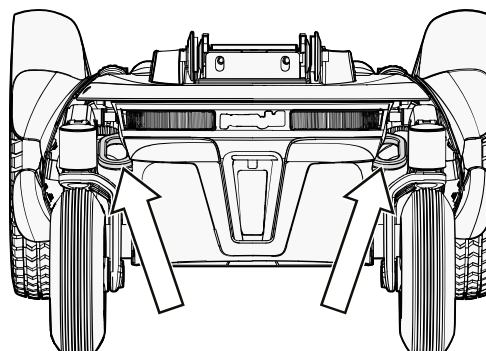


Figure 168. Rear tie-down eyes.

**DANGER!****User transportation in wheelchair**

When the user is seated in the wheelchair during transport, the following requirements must be met:

- The vehicle must be equipped with a locking system approved by Permobil for the product concerned. For alternative vehicle securement options, please consult your dealer.
- The locking system must be dimensioned for the total weight of the wheelchair.
- The user must use a three-point seat belt attached to the vehicle.
- The wheelchair must be crash tested and approved.
- The wheelchair must be fitted with a headrest.
- Alterations or substitutions should not be made to wheelchair securement points, structural parts, frame parts or components without consulting Permobil or the dealer.
- Spill-proof sealed batteries, such as gel or AGM, must be used.
- Make sure that wheelchair components do not accidentally press down the seat belt's release button.

If these requirements are not followed, the user can sustain bodily injuries or die.

- i** Permobil recommends that the wheelchair user is transferred to a vehicle seat and that he or she uses a factory-installed three-point seatbelt.

**WARNING!****Remove installed trays**

To reduce the risk of injury to the user and other vehicle occupants, wheelchair-installed trays that are not specifically designed for crash safety should:

1. be removed and secured separately in the vehicle, or
2. be secured to the wheelchair but positioned away from the user with energy-absorbing padding placed between the tray and the user.

**DANGER!****Risk of injury – correct seatbelt positioning**

The correct position for the vehicle's three-point seatbelt is on the inside of the wheelchair armrest. The three-point seatbelt should fit snug to the user's body without anything in the way. Failure to correctly position the three-point seatbelt may cause injury and/or death, if the vehicle is in an accident.

**WARNING!****Requirements for vehicles used for transportation**

The wheelchair may only be transported in vehicles approved for this purpose. Make sure the vehicle is suitably designed and equipped to transport a person in a wheelchair and that the tie-down points on the transporting vehicle are well-anchored.

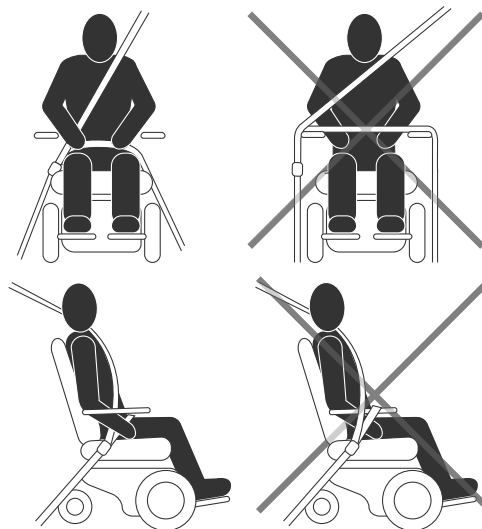


Figure 169. The illustrations on the left show correct positioning of the seatbelt positioning.

**CAUTION!****Damage during transportation**

You must inform Permobil as soon as possible after the event if the wheelchair and its accessories have suffered transportation damage, damage during driving or damage from other causes. There is a risk that the wheelchair and its accessories can no longer be used safely and securely. Contact your service provider or Permobil for further information.

**WARNING!****Positioning belts and aids are not vehicle seatbelts**

The wheelchair positioning belt and aids are designed to properly position the user in the wheelchair for safely operating the wheelchair. The wheelchair positioning belts and aids do not provide protection in a traffic accident and do not replace a vehicle mounted seatbelt.

**WARNING!****Use installed positioning aids**

The user must use and securely fasten the positioning belt and any other positioning aids on the wheelchair. Failure to properly use positioning belts and other positioning aids may cause bodily injury. If there is any sign of damage or wear on any positioning belt or aids, immediately contact Permobil for a replacement.

**WARNING!****Secure loose objects during transportation**

Auxiliary wheelchair equipment, loose or mounted on the wheelchair, must either be secured to the wheelchair or removed from it. Dismantled or movable auxiliary equipment must be properly secured in the vehicle during transit. This is to prevent loose parts or parts that may come loose from causing injury to the occupants during transit.

**WARNING!****Securing the wheelchair in vehicles**

The wheelchair may only be secured in position using approved fastening straps or a locking system. The locking system must be approved for the Permobil product concerned.

Secure the wheelchair by attaching approved fastening straps to the eyes at the front and rear, marked with yellow stickers. Do not attach the fastening straps to any other part of the wheelchair. Secure the fastening straps to the vehicle according to instructions for the restraint system in the vehicle.

Always make sure that the tie-down points in the transporting vehicle are well-anchored.

Check that the brake release on the wheelchair is set to locked position.

6.4.2 Air transportation



NOTICE

Preparations for air transportation

Airlines have different rules regarding wheelchair transportation. Please contact your specific airline for more information and to make sure the wheelchair can be transported safely.

6.4.2.1 Batteries

Gel batteries: in most cases, gel batteries do not have to be removed from the wheelchair.

The main circuit breaker must always be in the Off position.

Acid batteries: most airlines require that batteries be removed from the wheelchair and transported in special boxes that the airline may provide.

6.4.2.2 The wheelchair's dimensions and weight

The wheelchair's weight and dimensions are important, depending on the size and type of airplane in which the wheelchair is to be transported. Always check with the airline what rules apply.

6.4.2.3 Preventing damage to wheelchair

Since the wheelchair will be put with other goods in a confined space during air the flight, it is important to take preventive measures to minimize transportation damage to the wheelchair.

Cover the control panel with soft, shock-absorbing material (foam plastic or similar) and turn the control panel in towards the backrest. Protect other protruding objects in a similar manner. Where possible, adjust parts to the folded position. Tape any loose cables to the seat or covers.

7 Maintenance and repairs

7.1	Frequency of maintenance and inspection.....	221
7.2	Tool bag	222
7.3	Cleaning.....	223
7.3.1	Metal surfaces	223
7.3.2	Plastics	224
7.3.3	Upholstery, cloth and vinyl	224
7.4	Battery replacement.....	225
7.4.1	Removing batteries.....	225
7.4.2	Installing batteries	233
7.5	Batteries storage	240
7.5.1	Short-term storage	241
7.5.2	Long-term storage.....	241
7.6	Wheels and tires	242
7.6.1	Drive wheels	242
7.6.2	Inflating tires	254
7.6.3	Casters.....	257

7.7	Main circuit breaker	264
7.8	Brake release	265
7.9	Positioning belt.....	265
7.10	Serial number labels.....	266
7.10.1	Serial number label on chassis.....	266
7.10.2	Serial number label R-net power module	267
7.10.3	Serial number label on the control panel	268

For the wheelchair to work well, it is important that it be used correctly and regularly maintained. A well maintained wheelchair lasts longer and has a lower risk of defects.

**CAUTION!****Maintenance by a qualified service technician**

Only qualified service technicians should perform the maintenance and repair specified in this manual. Read all instructions carefully before proceeding. If any questions arise, contact Permobil for assistance.

**NOTICE****When replacing batteries or circuit breakers,
switch off main circuit breaker**

The main circuit breaker must always be switched off when batteries and fuses are replaced. Always switch off the power supply to the control panel before interrupting the power with the main circuit breaker.

**NOTICE****Additional tools may be required**

Some repairs may require tools other than those supplied with the wheelchair.

7.1 Frequency of maintenance and inspection

Permobil recommends compliance with the following maintenance and inspection schedule. Contact your authorized dealer for all service-related needs or questions.

Maintenance and inspection schedule	Daily	Weekly	Monthly	Yearly
Check battery level indicator and charge if necessary.	✗			
Check that the joystick panel and other control panels are not damaged.	✗			
Make sure all removable parts are securely fastened.	✗			
Check positioning belts for wear and make sure buckles work.	✗			
Check tires and inflate as necessary.		✗		
If the wheelchair is equipped with lights and turn signals, make sure they are operational and clean.		✗		
Clean the wheelchair and upholstery.			✗	
Check upholstery, seating, headrest, arm pads and calf pads for wear.			✗	

Maintenance and inspection schedule	Daily	Weekly	Monthly	Yearly
Check that the brake release and the brake release lever work properly.			X	
Complete inspection, safety check and service performed by an authorized wheelchair dealer.				X

7.2 Tool bag



The wheelchair comes with a tool bag with the following contents for use for maintenance and minor repairs.

i Other tools may be supplied depending on wheelchair configuration.

Tool	Area of use
Allen keys.	General maintenance and seat adjustment.
11, 13 mm wrenches.	General maintenance and replacing batteries.

7.3 Cleaning

Regular care and maintenance will prevent unnecessary wear and damage to your Permobil product.

-  Use only the following cleaning methods.
-  In the case of severe soiling of the upholstery or damage to the surface finish, contact Permobil for information.



CAUTION!

Switch off the wheelchair before cleaning

Always switch off the power supply to the wheelchair before cleaning.

7.3.1 Metal surfaces

Due to the high quality powder coating, optimum corrosion protection is guaranteed. Ideally, use a soft cloth or sponge, hot water and a mild detergent for normal cleaning. Wipe down carefully with a cloth and water, and dry off.

Remove scuff marks from semi-matte surfaces with soft wax (follow manufacturer's instructions).

Remove scuff marks and scratches from shiny surfaces using car polish, either liquid or paste. After polishing, apply soft car wax to restore the original surface gloss.

7.3.2 Plastics

For normal cleaning, wash plastic surfaces with a soft cloth, mild detergent and hot water. Rinse thoroughly and dry with a soft cloth. Do not use solvents or abrasive kitchen cleaners.

7.3.3 Upholstery, cloth and vinyl

For normal cleaning, wash the upholstery with lukewarm water and a mild, non-abrasive soap. Use a soft cloth or brush. Before the surface dries, wipe off any water or soapy water residues with a clean, dry cloth. Repeat this procedure to remove stubborn dirt or stains. Ink spots can sometimes be removed by washing with soap and water followed by isopropyl alcohol.

Do not use any cleaning method that is not listed above; other methods may attack and degrade the vinyl and may void the wheelchair's warranty.

If necessary, the cover may be removed before cleaning. See also the washing instructions on the upholstery materials.

7.4 Battery replacement

7.4.1 Removing batteries



WARNING!

Use safety gloves and safety goggles

Always use safety gloves and safety goggles when working with batteries. Exercise caution when using metallic tools or other objects while working with batteries. Batteries are heavy and charged devices and must be handled with great caution. Failure to follow any of these warnings could cause a short circuit, explosion, property damage and/or bodily harm.



CAUTION!

Recycling batteries

Used batteries must be disposed of responsibly in accordance with local recycling regulations.

1. Place the wheelchair on a level surface. If possible, raise the seat lift halfway up, to facilitate removal of the chassis top cover.
2. Switch off the power supply using the On/Off key on the control panel and switch the automatic main circuit breaker to OFF.

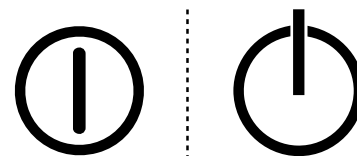


Figure 170. On/Off symbol depending on model.

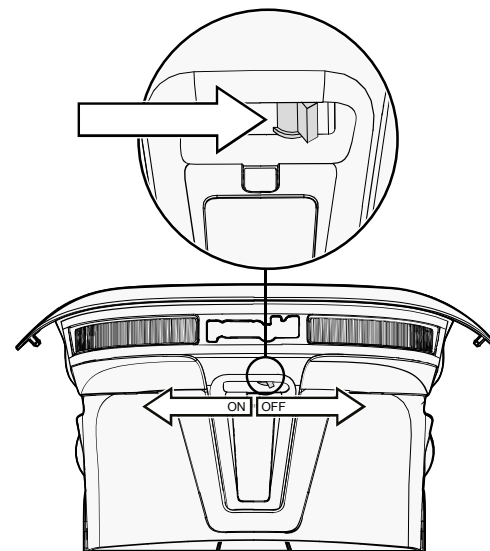


Figure 171. Main circuit breaker.

3. Remove the two knobs holding the chassis top and front covers.

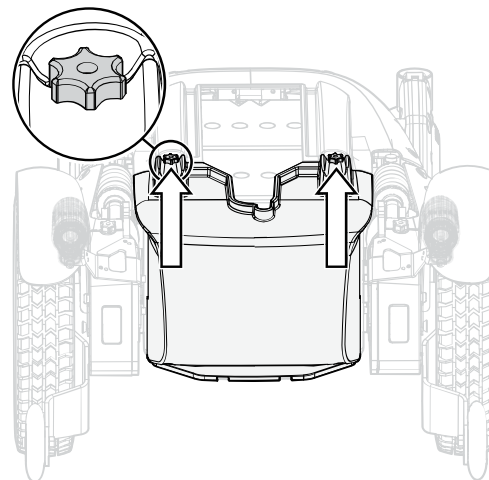


Figure 172. The chassis covers are secured with two knobs.

4. Slide the top cover off the chassis.

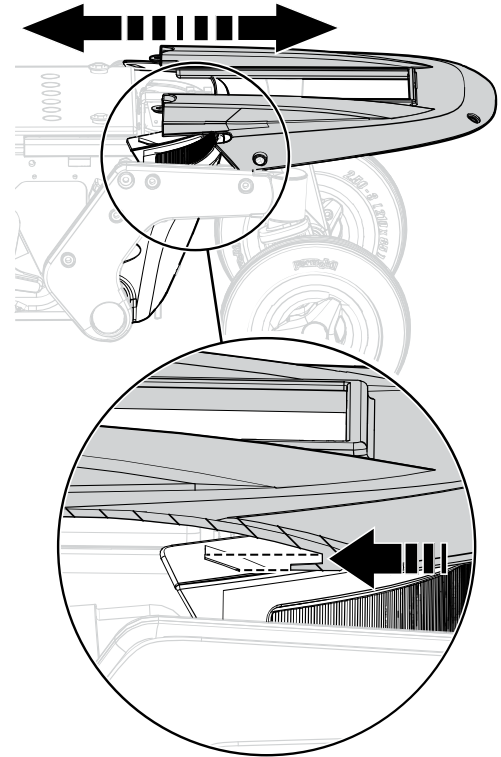


Figure 173. Top cover.

5. Pull the back cover off the dual locks and off the chassis.

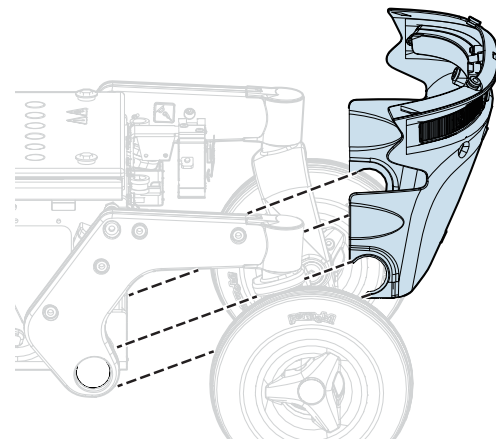


Figure 174. Back cover.

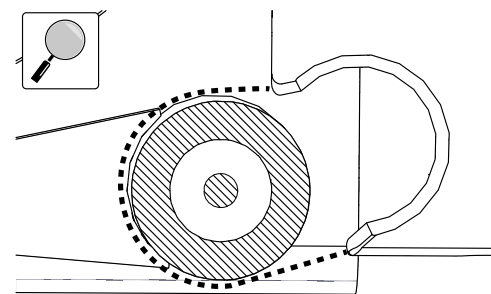


Figure 175. An enlargement of the back cover going over the rear axle.

**WARNING!****Do not load the seat**

Do not load the seat or the AP elevator during this operation. Any load on the seat or the AP elevator could cause permanent damage to the wheelchair or injuries on person(s) in the wheelchair or in its close vicinity. These conditions apply until the screws are reinstalled and tightened to the correct torque.

6. Remove the four screws holding the battery box.
7. Disconnect the right motor and inhibit cable connector (C).
8. Disconnect the left motor and the bus cable connector (B).
9. Disconnect the control panel connector (A).

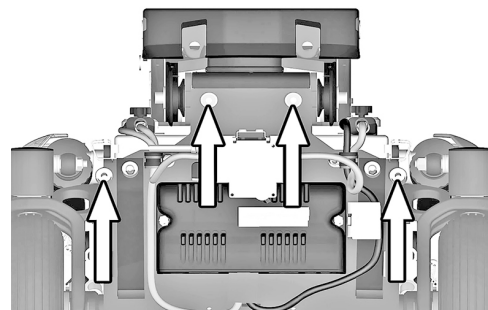


Figure 176. The battery box is secured with four bolts.

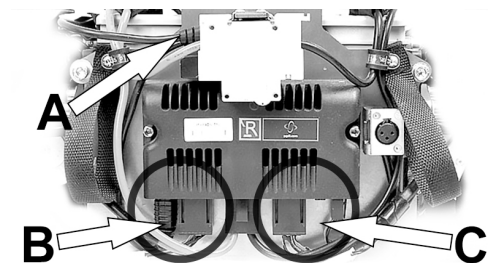


Figure 177. Connectors (B) and (C); for disconnection.

10. Use the straps to pull the battery box out of the chassis.


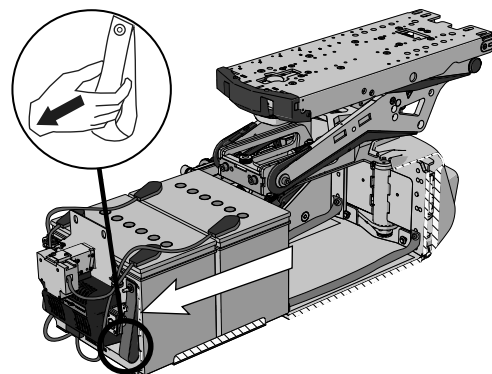


Figure 178. Straps for pulling out battery box.

11. Slide the battery terminal covers along the cables to access all four battery terminal screws.
12. Disconnect the cables from the four battery terminals.
13. Lift the batteries out of the battery box using the battery straps.

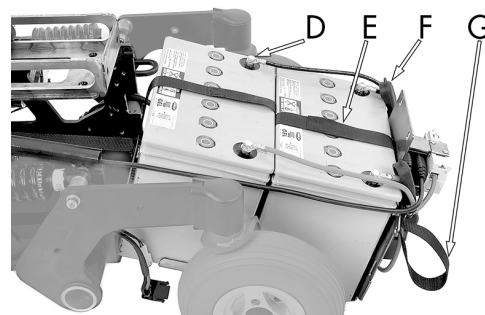

- D. Battery terminal.
- E. Battery strap.
- F. Battery terminal cover.
- G. Battery box pull-out strap.

Figure 179. Battery box when pulled out from chassis.

7.4.2 Installing batteries



NOTICE

Different types of batteries

The chair can be equipped with 60 Ah or 73 Ah maintenance-free batteries. Check carefully which battery you have.




CAUTION!

Always use recommended batteries

Always use Permobil recommended batteries. Other replacement batteries have not been tested for use with Permobil wheelchairs.

1. Use the battery straps and lift the new batteries in reverse order (leave the straps on the new batteries).

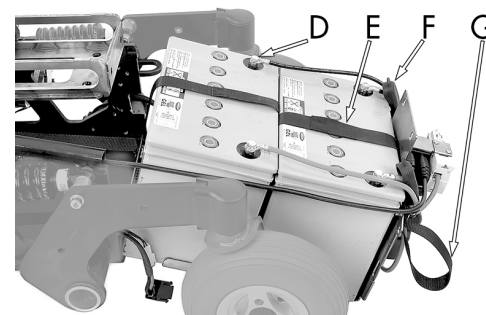

- D. Battery terminal.
- E. Battery strap.
- F. Battery terminal cover.
- G. Battery box pull-out strap.

Figure 180. Battery box when pulled out from chassis.

2. Make sure the batteries are positioned correctly to bring terminals into the right position; refer to the wiring diagram. Connect the four wires to the correct terminals on the batteries as shown in the diagram. Also refer to the sticker inside of the cover.

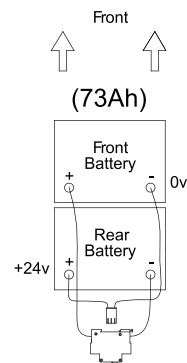


Figure 181 . Wiring diagram for the battery connection.

3. Attach the cable that is connected to the rear battery's left terminal on its cable holder.

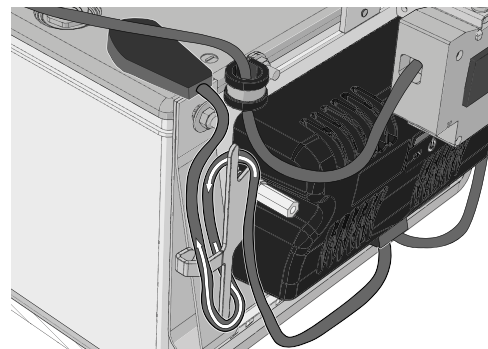


Figure 182. Attach the cable that is connected to the rear battery's left terminal on its cable holder as shown.

4. Attach the cable that is connected to the front battery's right terminal in its cable holder.

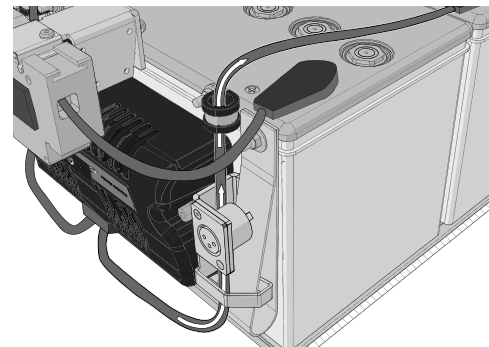


Figure 183. Attach the cable that is connected to the front battery's right terminal in its cable holder as shown.

5. Push the battery box in to the chassis.

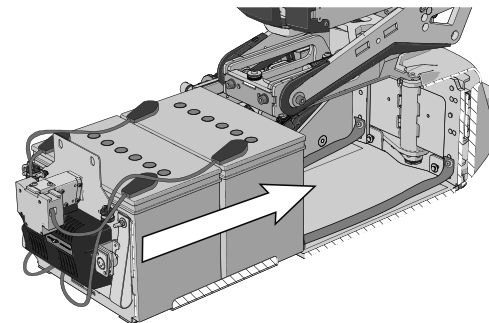


Figure 184. Push in the battery box.

6. Connect the right motor and inhibit cable connector (C).
7. Connect the left motor and bus cable connector (B).
8. Connect the control panel connector (A).

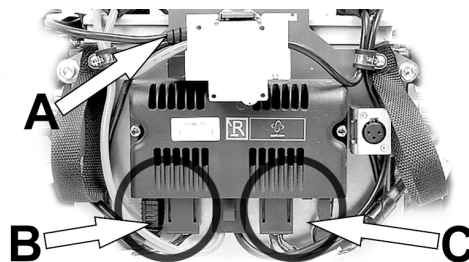


Figure 185. Connectors (B) and (C); for connection.

9. Refit the four screws securing the battery box. Use a torque wrench to tighten the screws. Tightening torque: 17.7 lb.ft.

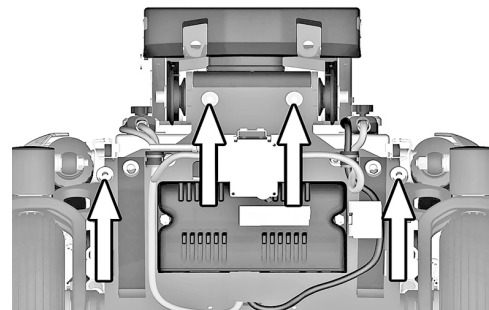


Figure 186. The battery box is secured with four bolts.

10. Refit the rear chassis cover on to the chassis.

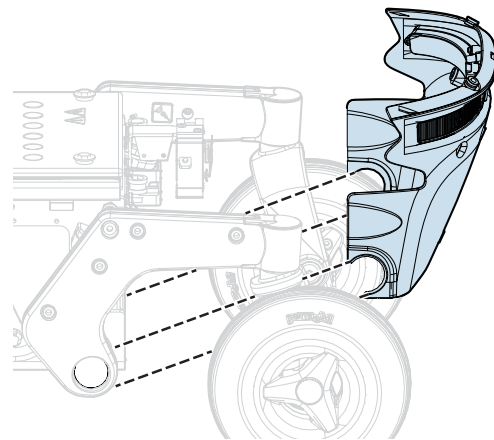


Figure 187. Rear cover.

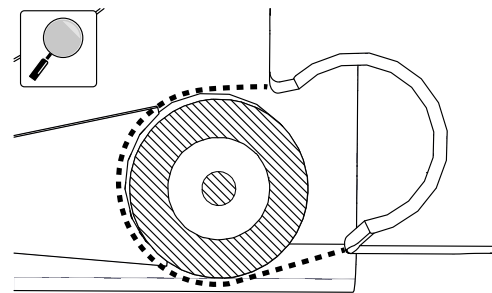


Figure 188. An enlargement of the rear cover going over the rear axle.

11. Refit the top chassis covers on to the chassis.

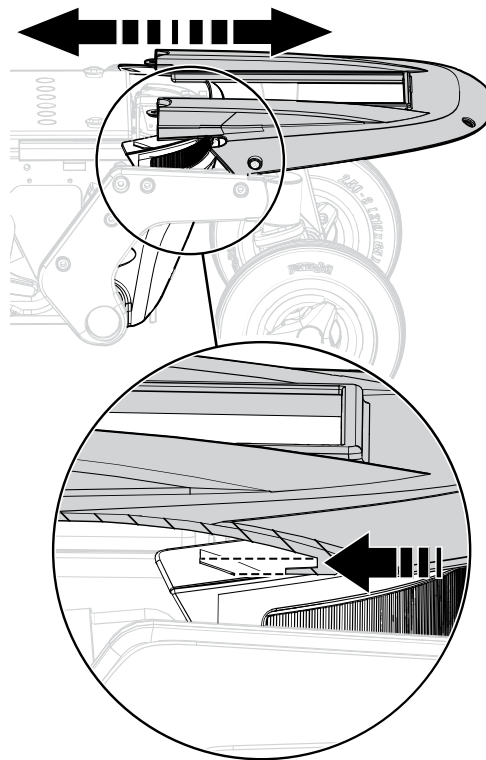


Figure 189. Top cover.

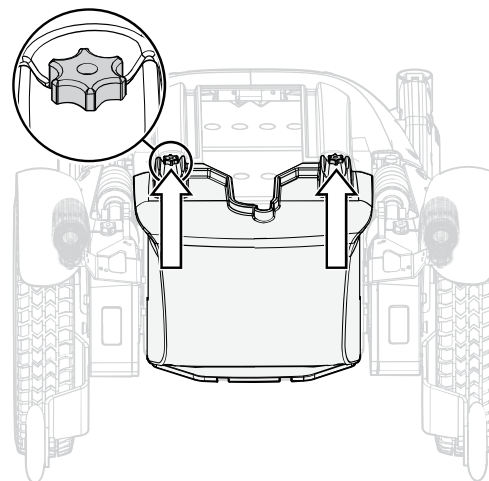
12. Refit the two knobs.

Figure 190. The chassis covers are secured with two knobs.

13. Switch the automatic main circuit breaker to the On position.

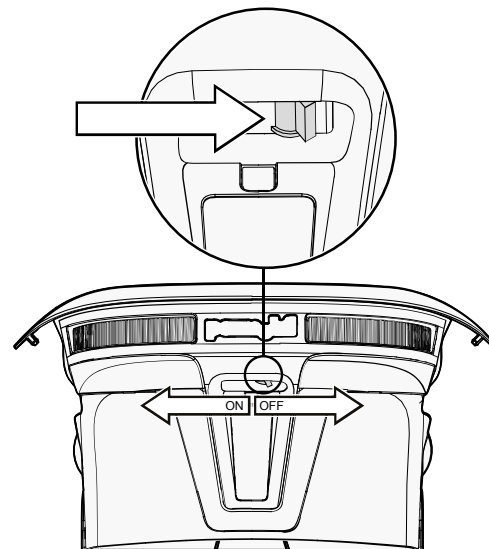


Figure 191 . Main circuit breaker located under the Permobil logotype on the rear cover.

7.5 Batteries storage

- The wheelchair must not be stored in areas subject to condensation (steam or moisture on surfaces), e.g. utility rooms or similar.

- If the wheelchair is equipped with acid batteries, the acid level must be regularly checked. If the wheelchair is equipped with gel or AGM batteries, the liquid level does not need to be checked.
- Battery service life depends on regular charging and avoiding complete discharge of the batteries.

7.5.1 Short-term storage

For short-term battery storage, the room must be at least 41.0°F. If the temperature is lower than this, the batteries may not charge fully and may be more vulnerable to corrosion.

7.5.2 Long-term storage

For long-term battery storage the room may be unheated, but to guard against battery corrosion, make sure the room is a few degrees warmer than its surroundings, as this will keep the room drier.

Switch off the main circuit breaker to avoid complete discharge of the batteries.

The batteries must be fully charged before storage. Batteries in storage require charging every 6 months.

7.6 Wheels and tires

7.6.1 Drive wheels

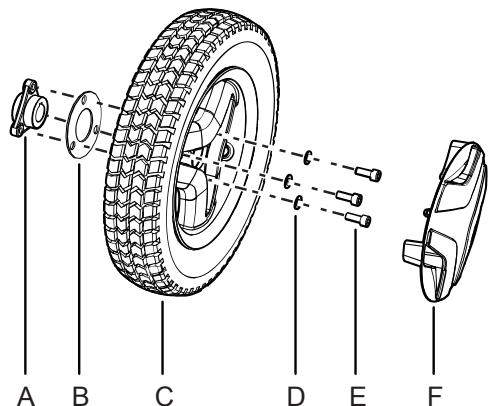


Figure 192. Installing the drive wheel.

- A.** Wheel hub, do not remove the hub from drive unit while performing service on the wheel.
- B.** Spacer, in use only when the wheelchair is fitted with winter tires.
- C.** Drive wheel.
- D.** Washers, ISO 7089 8 200 HV Fe/Zn 5 C1 (8,4x16x1,6).
- E.** Screws, ISO 4762 M8x20 8.8 Fe/Zn 5 C1/ TUF-LOK DIN 267-28.
- F.** Hub cap (the design may vary depending on markets and market regulations).

The following tools are necessary for this task:

- 1 Torque wrench.
- 1 Allen key socket, 6 mm.
- 1 Jack.
- 4 Blocks for securing the wheelchair.



NOTICE

Replace used wheel bolt

If a wheel bolt is removed for tire service, replace it with a new, unused bolt from Permobil and tighten it to the recommended torque. Also, inspect the drive axle and wheel rim for any damage. Damage to either part can cause the wheel bolt to loosen or fracture. Because the TUF-LOK thread lock fluid wears off, Permobil recommends that wheel bolts only be used once.

7.6.1.1 Removing the drive wheels



Do not remove the wheelhub (F) from drive unit while performing service on a wheel.

1. Switch off the main power switch on the control panel.

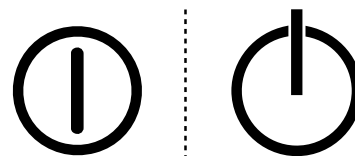


Figure 193. On/Off symbol depending on model.

2. Jack up the wheelchair until the wheel turns freely.

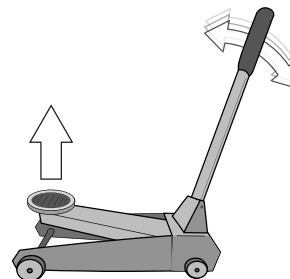


Figure 194. Use a jack or equivalent to lift up the wheelchair.

3. Use the blocks to secure the chair further.

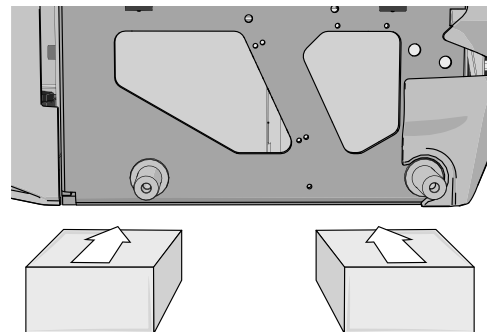


Figure 195. Use two blocks on each side of the chassis. The wheels have been removed in this figure for a better view.

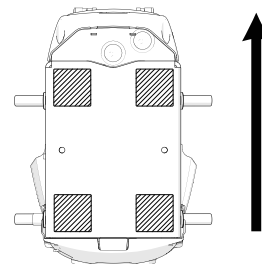


Figure 196. Block location. The arrow shows the direction of travel.

4. Remove the hub cap (the design may vary depending on markets and market regulations) by carefully levering it out using fingers on two edges of the hub cap.

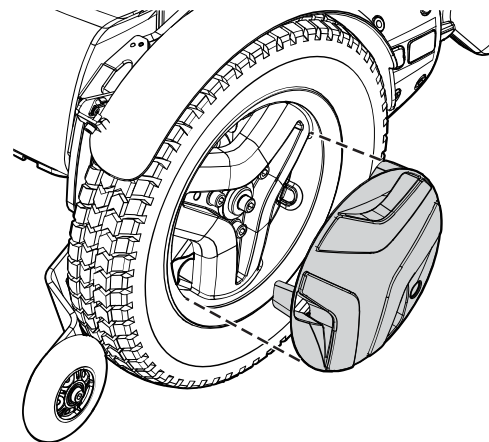


Figure 197. Use your fingers as follows on two edges of the hub cap.

5. Remove the three screws that hold the wheel in place. The central screw must not be removed.
6. Remove the wheel by pulling it straight out.
7. Remove the spacer (only on some models).

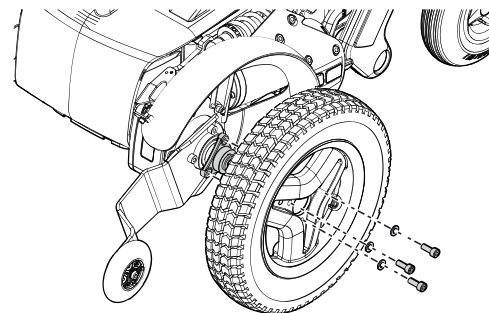


Figure 198. Pull the wheel straight out after you have removed the three screws.

7.6.1.2 Installing drive wheels

1. Fit the spacer (only on some models).
2. Fit the wheel onto the wheel hub.
3. Insert the three screws and the three washers. Tighten the screws no more than 11 lb.ft.

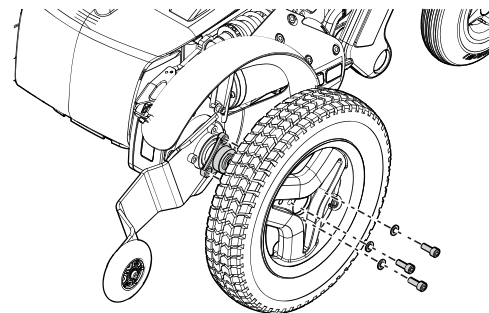


Figure 199. Fit the wheel onto the wheel hub.

4. When all screws and washers are in place, tighten the screws.
Tightening torque 17.7 lb.ft.

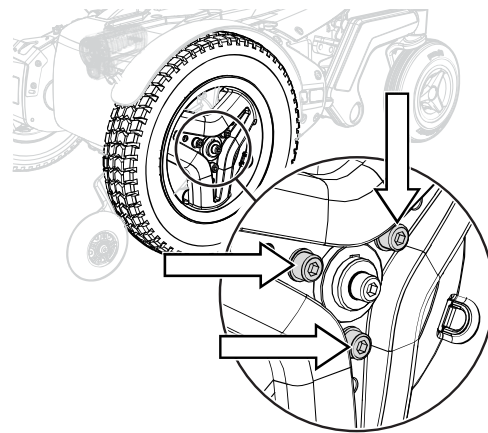


Figure 200. Use a torque wrench to tighten the screws.

5. Push the hub cap (the design may vary depending on markets and market regulations) in place.
6. Remove the blocks.
7. Lower the wheelchair using a jack or equivalent.

7.6.1.3 Drive wheel rim

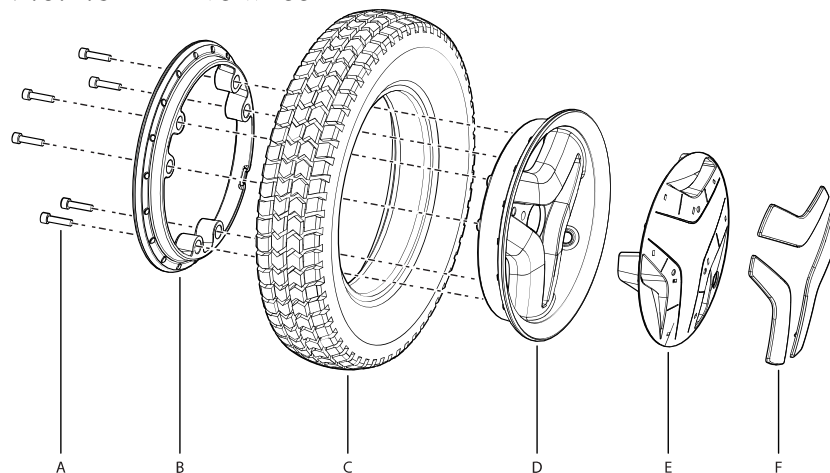


Figure 202. Fitting or removal of solid or pneumatic tires.

- A.** Screw, ISO 4762 M6x25 8.8 Fe/Zn 5 C1/TUF-LOK DIN 267-28.
- B.** Rim, inner section.
- C.** Tire (Inner tube only on pneumatic tires).

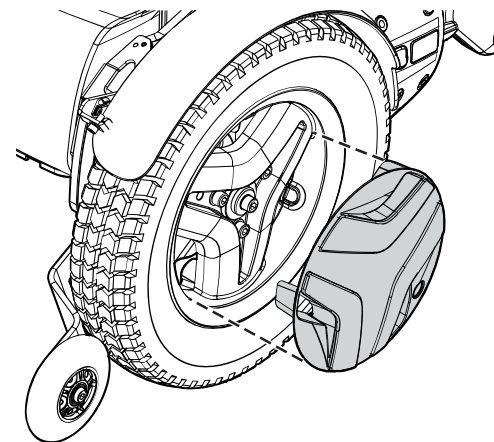


Figure 201. The hub cap snaps when it is in place.

- D.** Rim, outer section.
- E.** Hub cap
- F.** Wheel accent covers

7.6.1.4 Taking the drive wheel rim apart



WARNING!

Risk of injury - release air from tire

Before taking the wheel rim apart, release air from the pneumatic tire. Failure to do so may cause damage to the tire, rim and/or bodily injury.

The rim can be taken apart to allow fitting or removal of solid or pneumatic tires.

1. Remove the wheel from the wheelchair. See 7.6.1 *Drive wheels*, Page 242.
2. If the tire is pneumatic, release the air.
3. Remove the six screws holding the two halves of the rim together.
4. Take the rim apart.

7.6.1.5 Assembling the drive wheel rim

- i** Read all warnings contained in this section before filling the tires. Failure to do so may result in injury to the user and damage to the wheelchair and other property and also void any warranty applicable to the wheelchair.

Assemble in the reverse order. Tighten the six screws using a torque wrench. Tightening torque: 16.2 lb.ft. Inflate the tire to the recommended tire pressure: 35 psi.



CAUTION!

Risk of injury if tire pressure is incorrect

Before operating the wheelchair for the first time and regularly thereafter, check that the tire pressure meets the specifications in this manual. Check the tire pressure when the wheelchair experiences a significant change in temperature or altitude. Incorrect tire pressure may cause the wheelchair to be less stable, less maneuverable and cause damage to the wheelchair and/or bodily injury.

**NOTICE****Risk of damage if tires are overfilled**

Do not overfill the tires. Overfilling may result in damage to the wheel assembly.

**NOTICE****Risk of reduced performance when tire pressure is insufficient**

Insufficient tire pressure may result in abnormal wear and a shorter driving range.

**CAUTION!****Maintenance by a qualified service technician**

Only qualified service technicians should perform the maintenance and repair specified in this manual. Read all instructions carefully before proceeding. If any questions arise, contact Permobil for assistance.

7.6.2 Inflating tires

i Read all warnings contained in this section before filling the tires. Failure to do so may result in injury to the user and damage to the wheelchair and other property and also void any warranty applicable to the wheelchair.

i Applies only if the wheelchair is fitted with pneumatic tires.

At regular intervals, check that the wheelchair's tires have the prescribed pressure between 35 psi. Incorrect tire pressure can impair stability and maneuverability, while extremely low tire pressure can cause abnormal wear as well as shorter tire life.

1. Unscrew and remove the plastic valve cap on the tire air valve.
2. Connect the compressed air nozzle to the valve and adjust the tire pressure to the prescribed level.
3. Install the plastic valve cap.

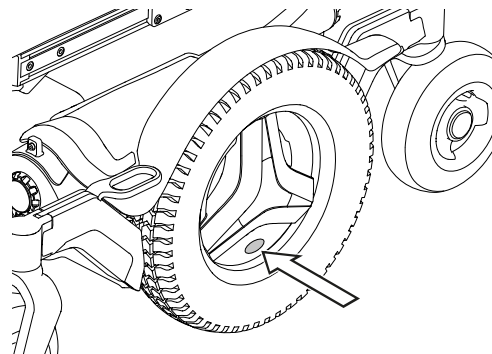


Figure 203. Filling valve on drive wheel.

**CAUTION!****Risk of injury if tire pressure is incorrect**

Before operating the wheelchair for the first time and regularly thereafter, check that the tire pressure meets the specifications in this manual. Check the tire pressure when the wheelchair experiences a significant change in temperature or altitude. Incorrect tire pressure may cause the wheelchair to be less stable, less maneuverable and cause damage to the wheelchair and/or bodily injury.

**NOTICE****Risk of damage if tires are overfilled**

Do not overfill the tires. Overfilling may result in damage to the wheel assembly.


**NOTICE****Risk of reduced performance when tire pressure is insufficient**

Insufficient tire pressure may result in abnormal wear and a shorter driving range.

**CAUTION!****Maintenance by a qualified service technician**

Only qualified service technicians should perform the maintenance and repair specified in this manual. Read all instructions carefully before proceeding. If any questions arise, contact Permobil for assistance.

7.6.3 Casters

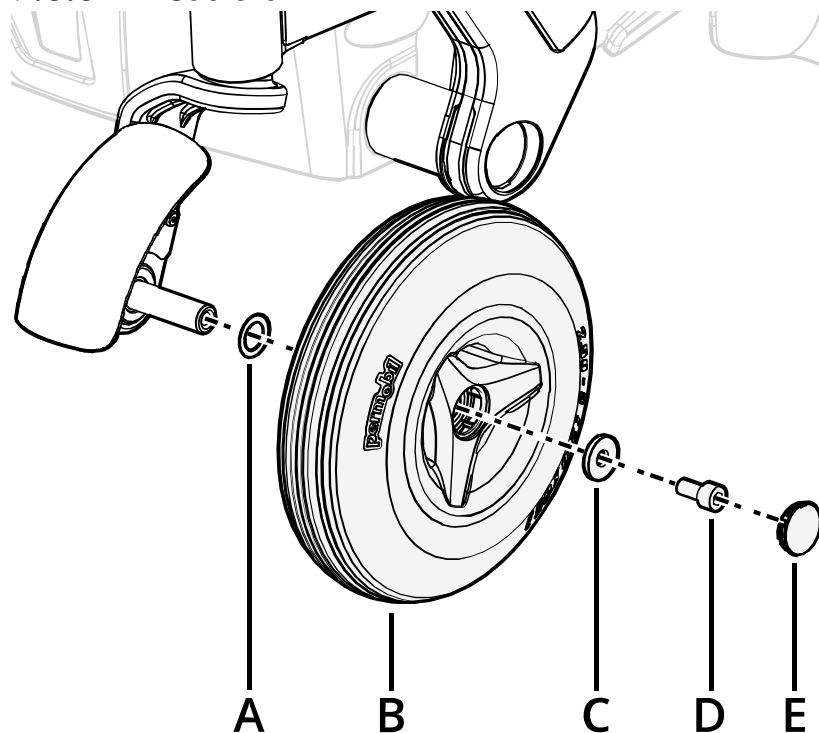

- A. Spacer.
- B. Wheel.
- C. Washer, 8,5x23x3.
- D. Screw, ISO 4762 M8x16 10.9 Fe/Zn/ TUF-LOK.
- E. Hub cap (the design may vary depending on markets and market regulations).

Figure 204. Assembling the rim.

The following tools are necessary for this task:

- 1 Torque wrench.
- 1 Allen key socket, 6 mm.
- 1 Jack.
- 4 Blocks for securing the wheelchair.

7.6.3.1 Removing casters

1. Switch off the main power switch on the control panel.

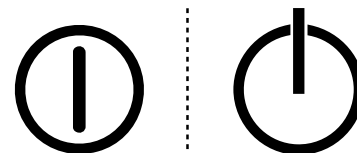


Figure 205. On/Off symbol depending on model.

2. Jack up the wheelchair until the wheel turns freely.

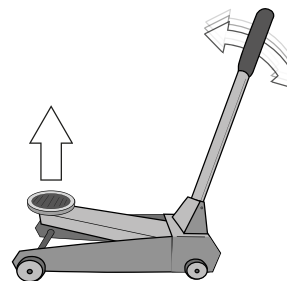


Figure 206. Use a jack or equivalent to lift up the wheelchair.

3. Use the blocks to secure the chair further.

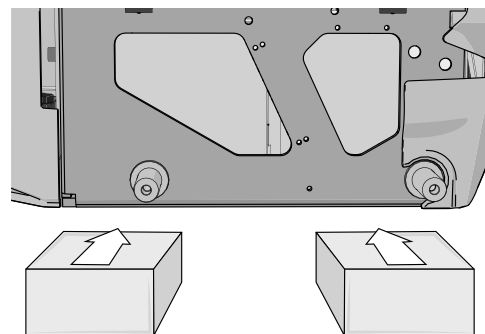


Figure 207. Use two blocks on each side of the chassis. The wheels have been removed in this figure for a better view.

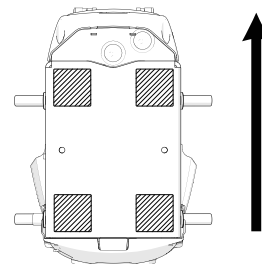


Figure 208. Block location. The arrow shows in the direction of travel.

4. Remove the hub cap (E) by carefully prying it out using a screw driver.
5. Remove the screw (D) and the washer (C).
6. Pull the wheel off the shaft.

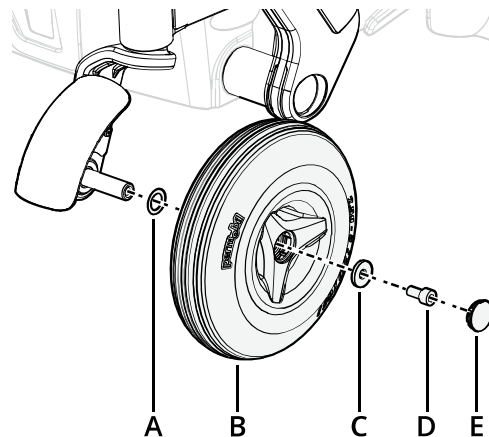


Figure 209. Assembling the rim.

7.6.3.2 Installing casters



NOTICE

Replace used wheel bolt

If a wheel bolt is removed for tire service, replace it with a new, unused bolt from Permobil and tighten it to the recommended torque. Also, inspect the drive axle and wheel rim for any damage. Damage to either part can cause the wheel bolt to loosen or fracture. Because the TUF-LOK thread lock fluid wears off, Permobil recommends that wheel bolts only be used once.

1. Check that the wheel shaft and rim are undamaged. Clean to remove dirt and rust. Replace damaged parts.
2. Fit the spacer (A) on the axle.
3. Install the wheel (B) on the axle by hand without using any tools. Make sure the rim is fully seated on the axle.
4. Use the screw (D) and washer (C) to install the wheel (B); do so by hand without using any tools.
5. Tighten the screw (D) using a torque wrench. Tightening torque: 17.7 lb.ft. Do not use a pneumatic impact wrench.
6. Fit hub cap (E).
7. Remove the blocks.
8. Lower the wheelchair using the jack.

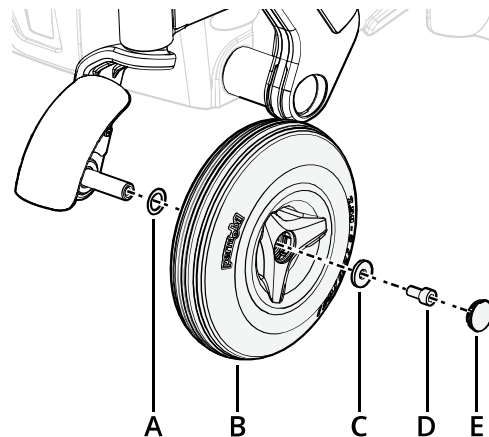


Figure 210. Assembling the rim.

7.6.3.3 Taking the caster rim apart

1. Remove the caster from the wheel fork. See 7.6.3.1 *Removing casters*, Page 258.
2. Remove the three bolts which holds the inner and outer parts of the rim together.
3. Take the rim apart.

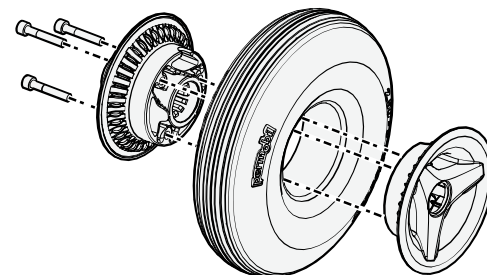


Figure 211. Rim.

7.6.3.4 Putting the caster rim together

1. Fit the two rim halves together with the tire.
2. Tighten the three screws using a torque wrench.
Tightening torque: 7.2 lb.ft.
3. Fit the wheel on to the wheelchair. See 7.6.3.2 *Installing casters*, Page 261.
4. Remove the blocks.
5. Lower the wheelchair with the jack or equivalent.

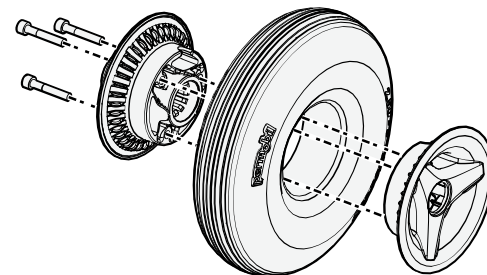


Figure 212. Rim.

7.7 Main circuit breaker

The main circuit breaker also functions as a battery isolator although it is referred to as the main circuit breaker in the user's manual.

The main circuit breaker is automatic and can be reset when it has tripped. Reset it by putting the switch to the On position.



NOTICE

Investigate tripped main circuit breaker

A tripped main circuit breaker often indicates a major electrical fault. The cause of a tripped main circuit breaker must be carefully investigated and determined before resetting the circuit breaker.



NOTICE

Before using the main circuit breaker

Always switch off the power to the control panel before switching off the main power with the main circuit breaker.

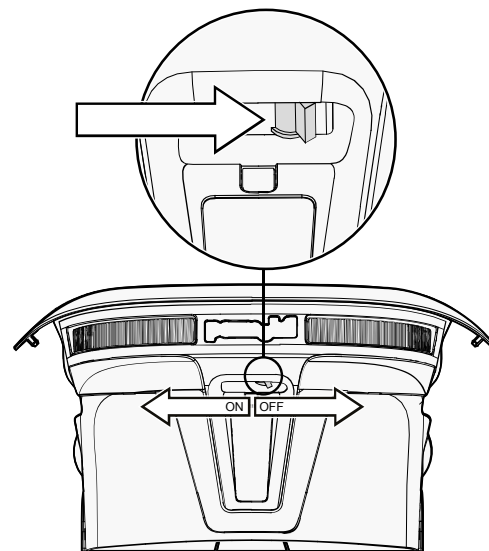


Figure 213. Main circuit breaker.

7.8 Brake release

Every month, check that the brake release and the brake release lever(s) are working properly.

When the brakes are released, it must not be possible to drive the wheelchair. If it can be driven, contact an authorized Permobil service center as soon as possible.

7.9 Positioning belt

i Accessory

Check the condition of positioning belts regularly in case any damage or wear has occurred. If signs of damage or wear appear, replace the positioning belt immediately through your Permobil dealer.

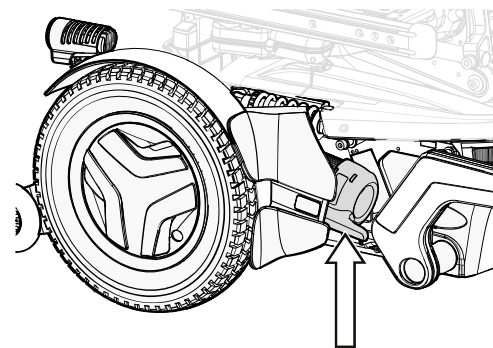


Figure 214. Brake release lever.

7.10 Serial number labels

7.10.1 Serial number label on chassis

The serial number label is located on the lower, left hand side of the wheelchair chassis. Look between the rim spokes.


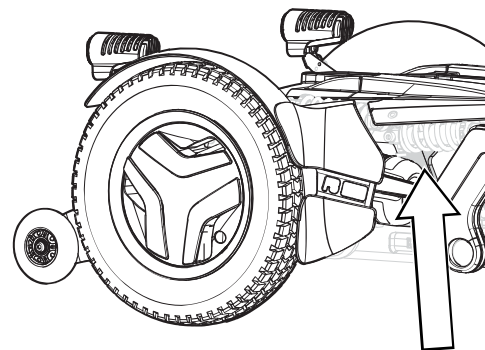


Figure 215. Chassis identification number label.

7.10.1.1 Serial number label description

- A. Made in (country of final assembly) by (address of site of final assembly).
- B. If sold by prescription a Rx-symbol will be on the serial number label.
- C. Maximum user weight.
- D. Model name of the product.
- E. Serial number.
- F. EAN code.
- G. Device identifier (01), Date of assembly (11), Serial number (21).

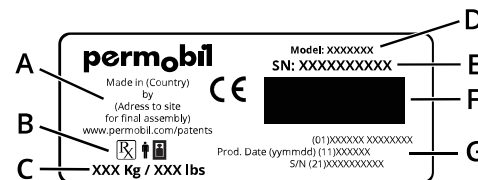

7.10.2 Serial number label R-net power module

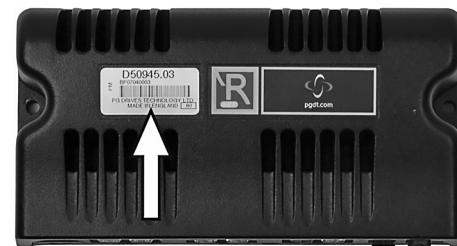


Figure 216. Power module ID number.

7.10.3 Serial number label on the control panel

The serial number label is only visible when the panel is removed from the panel holder.



Figure 217. Control panel ID number.


8 Restrictions and limitations


8.1	Support wheels	270
8.2	Speed restriction conditions	270
8.3	Conditions that limit seat movement	273
8.3.1	Actuator end limits excluded.....	273


8.1 Support wheels

The need for support wheels is based on seat depth, maximum speed, and user weight. If your wheelchair is not fitted with support wheels and your weight increases to more than lb., support wheels must be installed. Contact your service technician.

8.2 Speed restriction conditions

 User weight up to 220 lbs.					
Speed restriction	When	or when	or when	or when	or when
Low speed	Seat elevated > 2"	Back angle $\geq 150^\circ$ or $< 90^\circ$ and seat elevated < 2"	Tilt angle from -6° to -10° and seat elevated < 2"	Leg rest angle > 150° (to horizontal plane)	
Extra low speed	Tilt angle $< -10^\circ$	Tilt angle from -1° to -10° and seat elevated > 2"	Back angle $\geq 150^\circ$ or $< 90^\circ$ and seat elevated > 2"	Leg rest angle > 150° and seat elevated > 2"	Footplate below ground clearance due to leg rest manually extended
Drive inhibit	Tilt angle $< -10^\circ$ if support wheels not down	Tilt angle $< -20^\circ$ if support wheels not down			

 User weight between 220 lb. and 297 lb.					
Speed restriction	When	or when	or when	or when	or when
Low speed	Seat elevated > 2"	Back angle $\geq 150^\circ$ or $< 90^\circ$ and seat elevated < 2"	Tilt angle from -6° to -10° and seat elevated < 2"	Leg rest angle > 150° (to horizontal plane)	
Extra low speed	Tilt angle $< -10^\circ$	Tilt angle from -1° to -10° and seat elevated > 2"	Back angle $\geq 150^\circ$ or $< 100^\circ$ and seat elevated > 2"	Leg rest angle > 150° and seat elevated > 2"	Footplate below ground clearance due to leg rest manually extended
Drive inhibit	Tilt angle $< -10^\circ$ if stand and drive is set to disabled	Tilt angle $< -20^\circ$ if support wheels not down			

 User weight over 297 lbs.				
Speed restriction	When	or when	or when	or when
Low speed	Seat elevated 2" – 6"	Back angle $\geq 150^\circ$	Tilt angle -6° – -10°	Leg rest angle $> 150^\circ$ (angle to seat plate)
Extra low speed	Seat elevated $> 6"$	Tilt angle $< -10^\circ$	Tilt angle -6° – -10° and leg rest angle $> 150^\circ$ (to seat plate)	Tilt angle -6° – -10° and seat elevated $> 2"$
Drive inhibit	Tilt angle $< -10^\circ$ if "stand and drive" is disabled	—	—	—

8.3 Conditions that limit seat movement

8.3.1 Actuator end limits excluded

Conditions > Function	When	or when	or when
Seat lift stopped	Tilt back angle >130° and rear of seat raised less than 5” when user weight >220 lbs	—	—
Tilt backward stopped	Tilt back angle ≥180° ★)	—	—
Tilt forward stopped (anterior tilt is when tilt <0°)	Tilt back angle >130° and rear of seat raised less than 5” when user weight >220 lbs	Tilt back angle ≤85° ★)	—

Conditions > Function	When	or when	or when
Recline stopped	Tilt back angle $\geq 180^\circ$	Tilt back angle $\leq 85^\circ$	—
Leg rest stopped	Total leg rest angle $\leq 93^\circ$ when anterior tilt $> -10^\circ$	Total leg rest angle $\leq 98^\circ$ when anterior tilt $< -10^\circ$	—

Observe ★) Tilt or anterior tilt may continue if 'pushed backrest' is selected.

9 Troubleshooting

9.1	Troubleshooting guide.....	276
9.2	Diagnostics R-net LCD control panel	277
9.2.1	General.....	277
9.2.2	Diagnostic screens	278
9.2.3	Example of a screen showing a system fault	280
9.3	Diagnostics R-net LED control panel	280
9.3.1	Battery voltage indicator	280
9.3.2	Steady	281
9.3.3	Slowly flashing red LEDs, 1–2 LEDs.....	281
9.3.4	Rapidly flashing, 1–10 LEDs	281
9.3.5	Example of error messages and remedies	283
9.4	Repairing defective units	285

9.1 Troubleshooting guide

The following troubleshooting guide describes a number of faults and events which may occur when you use the wheelchair, together with suggested remedies. Note that the guide cannot describe all the problems and events which may occur and you should always contact your service provider or Permobil in case of doubt.

Event	Possible cause	Remedy
The wheelchair does not start.	Batteries discharged.	Charge the batteries.
	The cable connection to the control panel has come loose.	Insert the cable in the control panel.
	Main circuit breaker switched to off position after e.g. battery replacement.	Reset the main circuit breaker. See page 264.
	Main circuit breaker tripped.	See page 264.
The wheelchair cannot be driven.	Battery charger connected.	Stop charging. Disconnect the charging cable from the wheelchair charger socket.
	Brake release activated.	Reset the brake release.
	Wheelchair locked.	Unlock the wheelchair.
The wheelchair switches itself off after a certain period of inactivity (1 - 30 min).	The electronics' energy saving mode has been activated.	Switch the wheelchair on again using the start button on the control panel.

Event	Possible cause	Remedy
The wheelchair stops while being driven.	The cable connection to the control panel has come loose.	Insert the cable in the control panel.
	Main circuit breaker tripped.	See page 264.
The wheelchair can only be driven at reduced speed.	Seat lift or seat angle raised too high. Applies only to power seat lift and seat angle.	Lower the seat lift or seat angle.
The wheelchair cannot be charged.	Main circuit breaker switched to off position after e.g. battery replacement.	See page 264.
	The charging circuit breaker has tripped.	Wait five minutes, the circuit breaker will automatically reset.

9.2 Diagnostics R-net LCD control panel

9.2.1 General

When an error or a fault occurs in the wheelchair's electronics, information about it is displayed in the control panel display. This information can then be used to diagnose where the error, or fault, occurred and its cause.

Troubleshooting and repairs must always be performed by qualified personnel with good knowledge of the wheelchair's electronics.

9.2.2 Diagnostic screens

9.2.2.1 Current diagnostic screen

When the control system's integrated protection circuits have tripped so that the control system can no longer operate the wheelchair, a diagnostic screen is displayed in the control panel display.

This indicates a system fault, i.e. R-net has detected a problem somewhere in the wheelchair's power system.



NOTICE

The diagnostic screen displays error occasionally

If the fault is in a module not currently in use, it may still be possible to drive the wheelchair, but the diagnostic screen will display occasionally.

Switch off the wheelchair and leave it switched off for a few minutes. Restart the wheelchair. If the fault persists, you must switch off the wheelchair and contact your service provider. Write down the information displayed in plain text in the control panel display and pass it on to your service provider.

Do not use the wheelchair until the problem has been remedied or you have received other instructions from your service provider.

**WARNING!****Performing diagnostics**

Diagnostics may only be performed by personnel with knowledge of the wheelchair's electronic control system. Incorrect or poorly performed repair works may make the wheelchair dangerous. Permobil accepts no liability for any personal injury or damage to the wheelchair and its surroundings that occur due to incorrect or poorly performed repairs.

**NOTICE****Unapproved replacement of parts**

If any part is replaced without approval from Permobil, the wheelchair warranty will become void. Permobil accepts no liability for any loss that occurs as a result of a control system component being opened, adjusted or modified without permission.

9.2.3 Example of a screen showing a system fault

9.2.3.1 Identified module

The system fault indicator is displayed on the screen when the control system module has detected a problem. The codes below indicate where the problem is located.

PM = Power module

JSM = Joystick module

9.2.3.2 Error message

The error message provides a brief description of the error type.

9.2.3.3 Error code

The four-digit code indicates which protection circuit has tripped.

9.3 Diagnostics R-net LED control panel

9.3.1 Battery voltage indicator

Each time the wheelchair is started, parts of its electronics are checked. When a fault occurs in these parts, it is displayed on the control panel battery voltage indicator and the indicator for speed or driving profile in the form of one or more flashing LEDs.

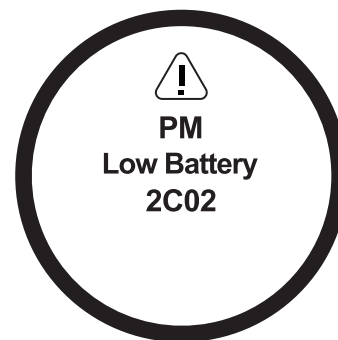


Figure 218. Screen showing system fault indication.

Troubleshooting and repairs must always be performed by qualified personnel with good knowledge of the wheelchair's electronics.



NOTICE

Error signals

Error messages are not displayed on the indicators while the wheelchair is being driven. They appear when it is next started.

9.3.2 Steady

Everything is in order. The number of LEDs that light up depends on the charge remaining in the batteries. If the batteries are fully charged, all the LEDs light up.

9.3.3 Slowly flashing red LEDs, 1–2 LEDs

The batteries must be charged immediately.

9.3.4 Rapidly flashing, 1–10 LEDs

A fault has been detected in the wheelchair's electronics and the wheelchair may not be driven.

1. Switch off the wheelchair.
2. Check that all visible cables and the cable to the control panel are connected correctly.
3. Switch the wheelchair on again. If the fault persists, count the number of flashing LEDs and check for a possible cause and remedy in the following table.
4. Do not use the wheelchair until the problem has been remedied or you have received other information from your service provider.

**WARNING!****Performing diagnostics**








Diagnostics may only be performed by personnel with knowledge of the wheelchair's electronic control system. Incorrect or poorly performed repair works may make the wheelchair dangerous. Permobil accepts no liability for any personal injury or damage to the wheelchair and its surroundings that occur due to incorrect or poorly performed repairs.



**NOTICE****Unapproved replacement of parts**

If any part is replaced without approval from Permobil, the wheelchair warranty will become void. Permobil accepts no liability for any loss that occurs as a result of a control system component being opened, adjusted or modified without permission.

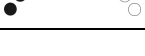9.3.5 Example of error messages and remedies

Event	Indication	Remedy
1 LED Low battery voltage		Check the condition of the batteries. Check the contact between the battery and the control unit.
2 LEDs Failure in left drive motor		Check the connection of the left drive motor.
3 LEDs Short circuit in left drive motor		Check the left drive motor's contacts and cables.

Event	Indication	Remedy
4 LEDs Failure in right drive motor		Check the connection of the right drive motor.
5 LEDs Short circuit in right drive motor		Check the right drive motor's contacts and cables.
6 LEDs Battery charger connected		Disconnect the battery charger.
7 LEDs Joystick error		Check that the joystick has not been moved when starting the wheelchair.
8 LEDs Control system error		Check the connectors between the drive unit and the power module.
9 LEDs Failure in brake circuit		Check the contacts to the magnetic brakes.
10 LEDs High battery voltage		Check the battery and the connectors between the battery and the power module.

Event	Indication	Remedy
7+5 LEDs Communication error		A communication error has been detected. Check that the cable to the control panel is not damaged and is correctly inserted.
Actuator indicator Actuator error		An actuator error has been detected. Contact authorized service for help.

9.4 Repairing defective units

Apart from specific OEM-approved spare parts, there are no replaceable parts in the R-net control system. Contact Permobil for further information on OEM-approved spare parts. Defective units must be sent for repair to Permobil or an authorized Permobil service center.

10 Electromagnetic compatibility

10.1	Electromagnetic emissions	288
10.2	Electromagnetic immunity	289
10.3	Recommended separation distance.....	293

10.1 Electromagnetic emissions

Guidance and manufacturer’s declaration

F5 Corpus VS and the battery charger (1825130) are intended for use in the electromagnetic environment specified below. The customer or the user of F5 Corpus VS and the battery charger (1825130) should assure that they are used in such an environment.

Emissions test	Compliance		Electromagnetic environment guidance
	F5 Corpus VS	Battery charger (1825130)	
RF emissions CISPR 11	Group 1	Group 1	F5 Corpus VS and the battery charger (1825130) use RF energy only for their internal functions. Therefore, their RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	Class B	F5 Corpus VS and the battery charger (1825130) are suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies power to buildings that are used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	Complies	

10.2 Electromagnetic immunity

Guidance and manufacturer's declaration

F5 Corpus VS and the battery charger (1825130) are intended for use in the electromagnetic environment specified below. The customer or the user of F5 Corpus VS and the battery charger (1825130) should assure that they are used in such an environment

F5 Corpus VS

Immunity test	Test level IEC 60601	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	If image distortion occurs, it may be necessary to position F5 Corpus VS further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.

Battery charger (1825130)

Immunity test	Test level IEC 60601	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.

Electrical fast transient/ burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	$< 5\% U_T$ ($> 95\%$ dip in U_T) for 0.5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $< 5\% U_T$ ($> 95\%$ dip in U_T) for 5 s	$< 5\% U_T$ ($> 95\%$ dip in U_T) for 0.5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $< 5\% U_T$ ($> 95\%$ dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the battery charger (1825130) requires continued operation during power mains interruptions, it is recommended that the battery charger (1825130) be powered from an uninterruptible power supply or a battery.
Power frequency (50/ 60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	If image distortion occurs, it may be necessary to position the battery charger (1825130) further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.

Note: U_T is the a.c. mains voltage prior to application of the test level.

F5 Corpus VS and the battery charger (1825130) are intended for use in the electromagnetic environment specified below. The customer or the user of F5 Corpus VS and the battery charger (1825130) should assure that they are used in such an environment

F5 Corpus VS			Electromagnetic environment guidance
Immunity test	Test level according to IEC 60601	Compliance level	Portable and mobile RF communications equipment should be used no closer to any part of F5 Corpus VS and the battery charger (1825130), including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = [3.5/V_1]\sqrt{P}$ $d = [3.5/E_1]\sqrt{P}$, 80 MHz-800 MHz $d = [7/E_1]\sqrt{P}$, 800 MHz-2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level. Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF IEC 61000-4-3	20 V/m 80 MHz-2.5 GHz	20 V/m	
Battery charger (1825130)			
Immunity test	Immunity test	Compliance level	
Conducted RF IEC 61000-4-6	3 V _{ms} 150 kHz-80 MHz	3 V _{ms}	
Radiated RF IEC 61000-4-3	3 V/m 80 MHz-2.5 GHz	3 V/m	



NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which F5 Corpus VS and the battery charger (1825130) are used exceeds the applicable RF compliance level above, F5 Corpus VS and the battery charger (1825130) should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating F5 Corpus VS or the battery charger (1825130).

10.3 Recommended separation distance

Guidance and manufacturer's declaration

F5 Corpus VS and the battery charger (1825130) are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of F5 Corpus VS and the battery charger (1825130) can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and F5 Corpus VS and/or the battery charger (1825130) as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter	Recommended separation distance according to frequency range of the transmitter				
	F5 Corpus VS		Battery charger (1825130)		
	80 MHz–800 MHz	800 MHz–2.5 GHz	150 KHz–80 MHz	80 MHz–800 MHz	800 MHz–2.5 GHz
0.01 W	0.018 m	0.035 m	0.12 m	0.12 m	0.23 m
0.1 W	0.056 m	0.11 m	0.37 m	0.37 m	0.73 m
1 W	0.18 m	0.35 m	1.2 m	1.2 m	2.3 m
10 W	0.55 m	1.1 m	3.7 m	3.7 m	7.3 m
100 W	1.8 m	3.5 m	12 m	12 m	23 m

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Index

A

Accessories	126
Admonitions	24
Armrest	165
Armrest, adjustment... 165–166, 169–170	

B

Backrest	162
Batteries, charging	201, 240
Batteries, location	60
Batteries, replacement	225
Batteries, storage	240
Battery voltage indicator	114
Bluetooth® mode	80
Bluetooth® mode, joystick functions	91
Brake release	188, 265
Buttons, ICS control panel	119

Buttons, Permobil Joystick Module for R-net	65
Buttons, R-net LED	111

C

Charger socket	62, 111
Chassis	58
Chest support	162
Chest support, adjustment ... 162–163	
Cleaning	223
ConnectMe	128
ConnectMe, airplane mode	136
ConnectMe, cleaning	135
ConnectMe, general use	135
ConnectMe, maintenance and service	135
ConnectMe, product approval ... 146	
ConnectMe, serial number label	149
ConnectMe, storage	135
ConnectMe, technical specifications	147
ConnectMe, turn off	144
Control panel, ICS seat	117
Control panel, R-net emergency stop	75
Control panel, R-net LCD diagnostics	277
Control panel, R-net LED	110
Control panel, R-net LED diagnostics	280
Control panel, R-net LED error message	283
Control system, R-net lock and unlock	103
Control system, R-net repair defective units	285
Cyber security	20

D

Display	70
---------------	----

Drive unit	58
Drive, downhill	197
Drive, general	186
Drive, obstacles	195
Drive, restrictions	193
Drive, slopes	196
Drive, technique	191
Drive, uphill	199

E

Electromagnetic compatibility	288
Electromagnetic interference	288

F

Footplate, adjustment.....	171–173
----------------------------	---------

H

Headrest, adjustment.....	156–161
---------------------------	---------

I

Important information	17
IR mode.....	93

J

Joystick panel, R-net LED	110
Joystick, error.....	200

K

Knee support, adjustment.....	174
-------------------------------	-----

L

Lights and reflectors	59
-----------------------------	----

M

Main circuit breaker.....	61
Main circuit breaker, reset	264

O

Ordering documentation.....	22
-----------------------------	----

P

Paddle switches, Permobil	
Joystick Module for R-net.....	65
Panel holder.....	178
Panel holder, parallel.....	180–182
Panel holder, rotational	179–180
Permobil Joystick Module for	
R-net	64
Positioning belt	265
Positioning belt, adjustment.....	183
Product approval	19

R

Reporting incidents	21
---------------------------	----

S

Safety instructions.....	23
--------------------------	----

Scrapping and recycling.....	22
Seat functions.....	51
Seat functions, power anterior tilt	121
Seat functions, power backrest angle.....	55, 120
Seat functions, power footplate ...	123
Seat functions, power leg length	121
Seat functions, power leg rest angle.....	56, 121
Seat functions, power seat lift.....	53, 120
Seat functions, power seat tilt.....	53, 120
Seat functions, power transfer	123
Seat-to-floor function	122
Serial number labels	266
Shock absorbers.....	58
Spare parts and accessories	21
Standing function	57, 122
Sticker, read instructions	150
Stickers.....	150

T

Technical specifications.....	46
Technical support.....	19
Tool bag.....	222
Transportation, air	215
Transportation, vehicle	207
Troubleshooting guide.....	276

W

Warnings and precautions.....	27
Warranty	18
Wheel locks	188
Wheels and tires	242
Wheels, casters	258
Wheels, drive wheels	242
Wheels, support wheels.....	152
Wheels, tires inflating.....	254

www.permobil.com

permobil



7 330818 354297

335340 eng-US