
Instructions for Use

TiLite[®] X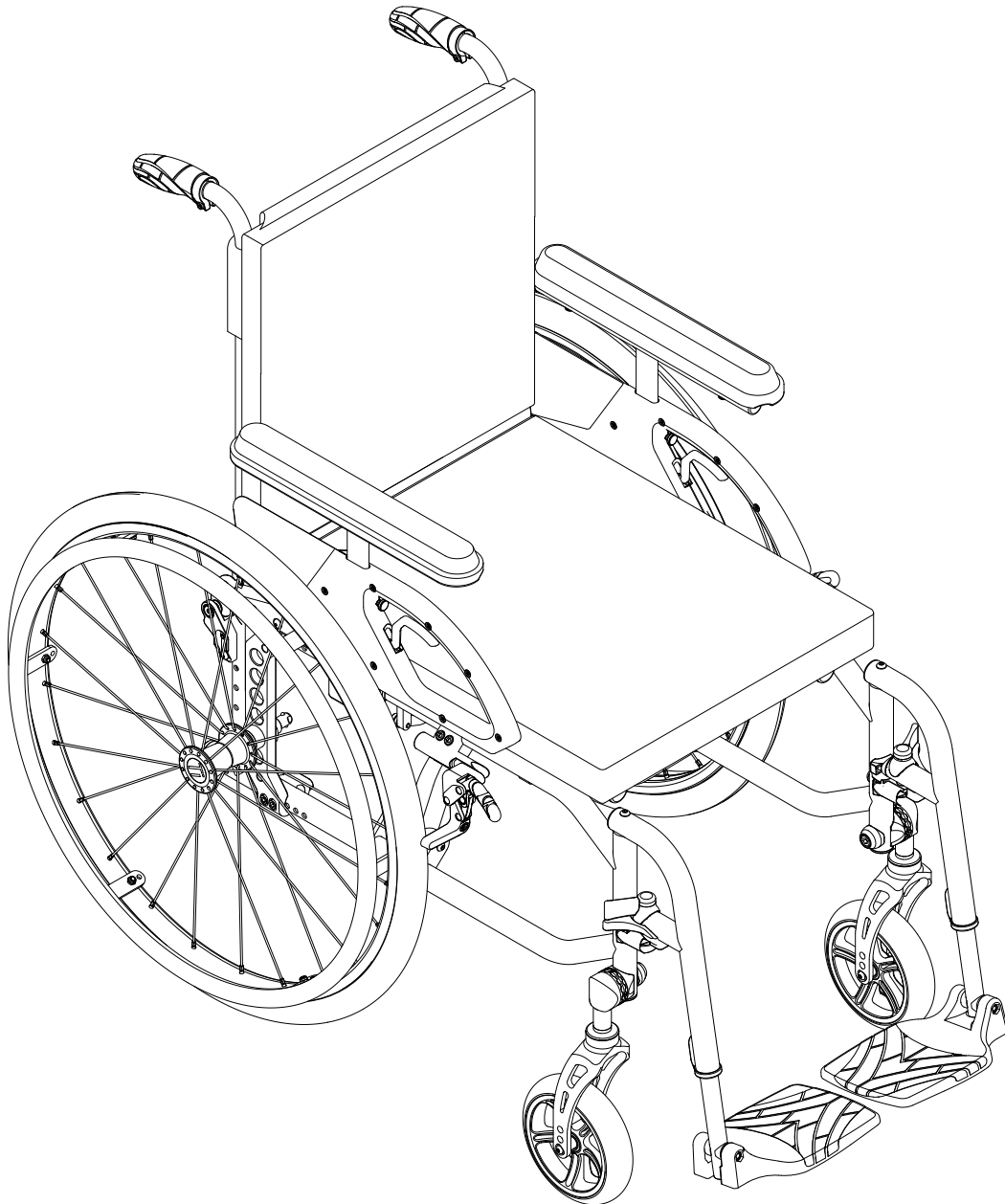

permobil

Contact Information

 TiSport, LLC
 2701 W. Court St.
 Pasco, WA 99301 USA
 Customer Support:
 U.S.A.:
 800-736-0925
 Fax: 866-586-2416
customerservice.tilite@permobil.com
permobil.com

 Warning

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

 Caution

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

Incident Reporting

In the event of a serious incident or injury, contact Customer Support.

Contents

Intended Use	2
Important Safety Information	2-4
Transportation	5-7
Product Specifications	8
Operation Instructions	9-14
Cleaning and Disinfecting	15
Troubleshooting	15
Maintenance	16
Storage, Disposal and Recycling	16
Español	17
Français	33

Warranty Information

Contact Permobil Customer Service or the dealer or distributor for information about the warranty period for this product. Information on applicable warranties can also be found at permobil.com.

Symbols Glossary

 Manufacturer	 Medical Device	 Consult instructions for use
 Date of manufacture	 Serial number	 Weight limit
 Unique Device Identifier	 Choking hazard	

Please Note

Dealer/Supplier: Please make sure the user of this wheelchair receives and understands the product information.

Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

One or more aspects of the products included in these instructions for use are patent pending, and are subject to one or more pending patent applications. In addition, products may be covered by one or more trademarks, including TiLite®.

Teflon® is a registered trademark of The Chemours Company.

Q'STRAIT® is a registered trademark of Valeda Company LLC.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

TiLite is part of Permobil.

© 2024 Permobil

The information detailed within these instructions for use apply to the Permobil TiLite X.

Please reference the relevant product owner manuals or knowledgeable equipment provider for any issues related to accessories not supplied by Permobil TiLite.

Intended Use

The intended use of this wheelchair is to provide mobility to physically impaired individuals. This manual wheelchair is intended for on-going “everyday” use and/or recreational sports applications.

The specific medical conditions for which the device is indicated are, but not limited to: Spinal Cord Injury, Post-Polio, Spina Bifida, and Amputee.

The maximum user weight for this wheelchair in the standard configuration is 300 lbs / 136 kg. The maximum user weight for this wheelchair in the heavy duty configuration is 350 lbs / 158 Kg. Exceeding the weight capacity of the wheelchair may cause tipping, instability, or damage to the wheelchair components.

Compatibility: It may be possible to configure the wheelchair with one or more other medical devices, accessories, or products. If a provider wishes to do so, a risk assessment should be performed by qualified equipment providers to make sure of the safety and efficacy of the combination. Third party accessories may interfere with the wheelchair's function, safety, or performance, and may void the warranty.

For information on proper mounting and configuration of a Permobil SmartDrive, please refer to the Permobil SmartDrive product documentation. If the wheelchair came configured with a SmartDrive, read the user manual for the SmartDrive prior to operating the wheelchair.

TiLite manufactures a wide variety of wheelchairs to meet the varied needs of wheelchair users. However, TiLite is not a healthcare advisor and does not know the needs and medical condition(s) of each user. The selection of a wheelchair, how it is adjusted, types of options, and accessories used is the responsibility of the user and a qualified medical professional.

The choice of a wheelchair and setup for safety depends on the following:

- *Disability, strength, balance and coordination of the user;*
- *Hazards of daily use (living and working environment and other places a user is likely to visit);*
- *Options needed for safety and comfort (anti-tips, positioning belts or other special seating systems, etc.).*

Failure to follow all considerations and safety information may result in serious bodily injury.

Important Safety Information

General Warnings

- DO NOT operate the wheelchair without first reading and understanding all of the provided instructions for use. Make sure that attendants read the instructions for use before assisting with the wheelchair. If unable to perform any tasks in the instructions for use, seek assistance by contacting a clinician, equipment provider, distributor, or Customer Support.
- Follow all instructions for use and safety information provided with the wheelchair and provided by the manufacturer of other accessories.
- Practice necessary activities of daily living and wheelchair skills in the presence of a qualified clinical professional before attempting active use of the wheelchair.
- The wheelchair should be evaluated annually by a qualified equipment provider, or when a change occurs in a user's physical or functional presentation or the performance of the wheelchair.
- DO NOT expose the wheelchair to high heat, open flames or hot ashes. Exposure to high temperatures may cause fire, burns, or damage to the wheelchair materials. Testing or certification claims, including for flammability, may no longer apply to the wheelchair when it is combined with other products or materials. Check testing and certification claims for all products used in combination with the wheelchair.
- To avoid permanent damage to the wheelchair, DO NOT use or leave the wheelchair in conditions of extreme heat or extreme cold.
- If planning to lift or carry heavy objects, make sure the objects do not exceed the listed maximum weight capacity of the wheelchair.
- DO NOT use the wheelchair as storage for other devices, especially heavy or sharp objects.
- DO NOT use the wheelchair in a roll-in shower.

Use of the Wheelchair

- If the wheelchair is configured with positioning aids (such as a pelvic positioning belt), make sure they are properly fastened prior to use of the wheelchair.
- If the wheelchair was configured with wheel locks, make sure they are properly engaged and able to maintain the wheelchair's position prior to use. DO NOT use wheel locks as dynamic braking devices for slowing forward motion.
- The wheelchair was designed to be used with a cushion on top of the seating support surface. DO NOT use the wheelchair without a proper cushion that has been installed to the cushion manufacturer's specifications. Make sure that the cushion is adhered properly to the seat sling of the wheelchair before use. Assessment and selection of a cushion should be performed by qualified clinical team and equipment provider.
- The wheelchair may be equipped with a tension-adjustable seat sling or back support. Proper tensioning of these components should be done by qualified equipment providers and maintained during use of the wheelchair.
- The wheelchair may be configured with anti-tip devices. Before using the wheelchair, make sure the anti-tips are correctly installed to prevent a rearward fall.
- DO NOT use the footplate as a platform when getting into or out of the wheelchair.
- DO NOT stand on the frame of the wheelchair.

Important Safety Information, continued**Use of the Wheelchair, continued**

- It is the responsibility of the equipment provider, therapist, and other qualified health care professionals to determine if seating and positioning accessories are required to make sure of the safe operation of this equipment by the user. Serious injury can occur in the event of a fall from a wheelchair.
- DO NOT place or store items on the cross brace. DO NOT lift the wheelchair by the cross brace as the wheelchair can collapse unintentionally.
- DO NOT lift the wheelchair by the hangers, side guards or armrests. These components can separate from the frame and the wheelchair can fall.
- If the wheelchair is equipped with a residual limb support, DO NOT use it to support weight during transfers.
- DO NOT allow feet to slip forward off the footplate during use. Feet should be securely positioned on the footplate with the appropriate angle and height adjustments. Allowing feet to slip forward off the footplate may cause injury to feet or legs.
- DO NOT use the armrests when the height adjustment is not fully engaged. The armrests should be adjusted to the appropriate heights and locked securely in place. Using the armrests when the height adjustment is not fully engaged may cause the armrests to collapse or detach and result in injury or equipment damage.
- Make sure that the side guards are fully engaged in their mounting brackets before use. Loose or detached side guards may cause injury or damage to you, the wheelchair, or the environment.
- DO NOT use the wheelchair if the user's body is too wide for the wheelchair. Using a wheelchair that is too narrow may cause side guard damage, injury or discomfort or damage to the wheelchair.

**Adjustments to the Wheelchair**

- The relative position of the footrest, center of gravity, rear seat height, caster, and back support are directly related to the wheelchair's stability. Any change to one or a combination of these or other adjustments on the wheelchair can cause the wheelchair to decrease in stability. Use extreme caution when using a new seating system or position for the first time.

**Navigational Concerns**

- DO NOT use an escalator to move the wheelchair between floors. Serious bodily injury may occur.
- DO NOT attempt to ride over curbs or obstacles. Doing so may cause the wheelchair to tip over and cause a fall.
- DO NOT use the wheelchair in inappropriate environments such as extreme hot or cold, wet or humid environments, or environments infested with pests. DO NOT use the wheelchair on the road or highway, or on surfaces that are uneven, slippery, or unstable such as grass, sand, loose soil, mud, or sticky terrain. Inappropriate environments may cause injury, damage, or malfunction of the wheelchair.
- DO NOT attempt to navigate inclines greater than 9 degrees, or that are too steep to adequately remain in control of the wheelchair.
- DO NOT attempt to ascend or descend stairs without the assistance of a competent caregiver.
- DO NOT attempt to navigate a corner, reverse, or stop too quickly. DO NOT perform wheelies or other stunts with the wheelchair. DO NOT use the wheelchair without spending enough time acclimating to it or if the center of gravity is adjusted too far back. Doing so may cause the wheelchair to tip over. Serious bodily injury may occur.

**Accessories, Spare Parts, and General Maintenance**

- Periodically, check the overall condition of the wheelchair to make sure it is in good operating condition. Look for loose or broken parts or fasteners, wear or damage to components that alter the functionality and safety of the wheelchair, and any rough edges that may cause injury.
- DO NOT use water-based lubricants on or around the front caster and fork as they can cause damage to the bearings.
- Solicit repair by a qualified individual and use only replacement parts manufactured by Permobil. Unauthorized repair or replacement parts may damage the wheelchair or void the warranty.
- DO NOT use the wheelchair unless it has the proper tire pressure (p.s.i.). DO NOT overinflate the tires. Failure to follow these warnings may cause the tire to explode and cause harm.
- Replacement of the tire or tube must be performed by an authorized Permobil dealer or qualified technician.
- DO NOT over-tighten hardware attaching any accessories to the frame. This could cause damage to the frame tubing or permanent damage to the threaded fastener or accessory.
- After making any adjustments, always make sure that parts are properly tightened before using the wheelchair.
- If the wheelchair is dropped, shipped, or stowed during air travel, carefully inspect the wheelchair to make sure it was not damaged prior to use.
- When preparing the wheelchair for use after side guards or armrests have been removed, make sure components are fully seated in their receivers prior to use.

Important Safety Information, continued

Cautions

- Exercise caution when transporting the wheelchair up or down stairs with an occupant in the wheelchair. Possible injury to the occupant and/or attendants can occur.
- When operating the wheelchair, take care to avoid getting fingers caught in the spokes of the spinning wheels.
- Use caution while tilting the wheelchair or otherwise lifting the front casters off the ground, as the risk of tipping backwards will increase.
- Use caution while operating the wheelchair on roads, streets or highways.
- Use caution while backing up in the wheelchair.
- Use caution while attempting to reach for objects on the ground, reaching too far forward, or attempting to dress or undress while sitting in the wheelchair, as these activities increase the risk of tipping over.
- Make sure the back support is locked securely in place and all attachment hardware is tight before operating the wheelchair.
- If the wheelchair was configured with quick release axles, make sure they are properly installed and seated prior to operating the wheelchair by attempting to remove the wheel after the quick release axle engages. If the wheel comes off, DO NOT operate the wheelchair until the quick release axles are properly adjusted.
- When transferring from the wheelchair, every precaution should be taken to do so safely and with minimal effort. Speak with a health care team for training on how to properly transfer from the wheelchair. Make sure both wheel locks are engaged to prevent the wheelchair from inadvertently moving backwards.
- Avoid placing full body weight on a single armrest, side guard or fender, or the residual limb support during a transfer or pressure relief.
- Make sure to keep the wheelchair clean by following the cleaning and disinfection instructions in this manual. Improper cleaning or disinfection may cause infection, damage, or malfunction of the wheelchair. Promptly clean any mud, sand, grass or other dirt from the wheelchair and allow to dry completely after cleaning. If the wheelchair becomes wet, dry the wheelchair as soon as possible and re-lubricate the caster axles, caster forks, rear wheel hubs and rear wheel axles.
- Make sure all detachable or small parts are handled with care. DO NOT lose or misplace the detachable parts of the wheelchair such as the armrests or cushions. Detachable parts are essential for the wheelchair's function, safety, and comfort. Losing or misplacing detachable parts may compromise the wheelchair's performance or cause injury.
- Keep small hardware components out of the reach of children and any individuals who have a tendency to place inedible objects in their mouth. Swallowing or inhalation may lead to serious injury or death. IMMEDIATELY seek emergency medical assistance. 
- Be careful when loading or unloading the wheelchair into or from a vehicle. DO NOT drop the wheelchair or handle it roughly. Dropping or mishandling the wheelchair may cause damage or malfunction of the wheelchair components.
- DO NOT wear different height shoes when using the wheelchair. Wearing different height shoes may affect the efficacy of the seating and positioning components on the wheelchair, resulting in possible injury or discomfort.
- Follow the recommendations of a qualified provider when adjusting the residual limb support. The residual limb support should be adjusted to provide optimal support and comfort for your residual limb. Improperly adjusted residual limb support may cause injury or discomfort to you or damage to the wheelchair.
- Be aware of the risk of damage during air travel. Follow the airline's instructions on how to prepare the wheelchair for air travel. Remove detachable parts and pack them securely. Label the wheelchair with the user's name and contact information. Check the wheelchair for any damage or missing parts after air travel.
- Make sure the armrests are adjusted to appropriate heights and that they are fully engaged in their mounting brackets before use. Improperly adjusted or secured armrests may cause injury or discomfort to you or damage to the wheelchair.
- Avoid pinching of fingers while actuating the residual limb support swing away release mechanism. The release mechanism should be operated carefully and smoothly. Pinching of fingers may cause injury or pain.
- Follow the proper shipping protocols found in this manual when sending or receiving the wheelchair. Use the original packaging or equivalent materials. Label the package with the appropriate information. Check the wheelchair for any damage or missing parts after shipping.
- Make sure that there are no obstacles in the cross brace area prior to folding the wheelchair.

Transportation

Occupied Use in a Motor Vehicle

Please read and understand all warnings before using the wheelchair. Safety is our top priority.

Warnings:

- This wheelchair conforms with the requirements of ISO 7176-19:2022 and has been designed and tested for use only as a forward-facing seat in a motor vehicle.¹
- The wheelchair has been dynamically tested in a forward-facing orientation with the ATD restrained by a three-point belt restraint.
- Only the designated securement points should be used to secure the wheelchair to the vehicle.
- This wheelchair has been tested with the Q-STRAINT Standard Lap Belt restraint.
- Both pelvic- and shoulder-belt restraints should be used to reduce the possibility of head and chest impacts with vehicle components.
- Pelvic- and shoulder-belt restraints should be used together as designed for (DO NOT connect a latch plate to a pin-bushing anchorage unless the system is designed to interface this way).
- In order to reduce potential of injury to vehicle occupants, wheelchair-mounted trays not specifically designed for crash safety should be removed and secured separately in the vehicle. If the tray cannot be removed, it should be secured to the wheelchair but positioned away from the occupant with energy-absorbing padding placed between the tray and the occupant.
- When possible, other auxiliary wheelchair equipment should be secured to the wheelchair or removed from the wheelchair and secured in the vehicle during travel so that it does not break free and cause injury to vehicle occupants in the event of a collision.
- Postural supports should not be relied on for occupant restraint in a moving vehicle.
- The wheelchair should be evaluated by a manufacturer's representative for determination whether the wheelchair is suitable for reuse after involvement in any type of vehicle collision.
- Alterations or substitutions should not be made to the wheelchair securement points or to structural and frame parts or components since this can affect the crashworthiness of the wheelchair, and it can also change the performance of the wheelchair in normal use. If it is considered necessary to make these kinds of alterations, the wheelchair manufacturer shall be consulted.
- Care should be taken when applying the occupant restraint to position the seatbelt buckle so that the release button will not be contacted by wheelchair components during a crash.

¹ Crash testing is a simulation of a frontal impact only. It does not simulate any other type of impact. Furthermore, TiLite wheelchairs are highly customized and can be ordered in millions of combinations and it is impossible to test every conceivable combination. Therefore, TiLite recommends that wheelchair users transfer to the vehicle seat when being transported in a vehicle. The vehicle seat offers the greatest degree of safety because it is secured to the chassis of the vehicle and is designed with the primary purpose of protecting the occupant in a crash. By contrast, the primary purpose of any wheelchair is to maximize mobility, which in turn requires that the product be as light as possible. As of this date, the U.S. Department of Transportation has not approved any tie-down system for transportation of a user while in a wheelchair in a moving vehicle of any type.

Transportation, continued

Transit Tie Downs

Wheelchair users should transfer to the vehicle seat and use the vehicle-manufacturer-installed restraint systems whenever it is feasible, and the unoccupied wheelchair should be stored in a cargo area or secured in the vehicle during travel. If the wheelchair must be occupied during vehicle travel, please ensure the following information is taken into account:

- The minimum recommended occupant mass for use in a transit equipped chair is 50 lbs (23 Kg).
- The maximum recommended occupant mass for use in a transit equipped chair is 300 lbs (136 Kg).
- The TiLite X with Transit Tie-Down Option has been found to meet or exceed the performance requirements of ISO 7176-19:2022. This standard requires that transit equipped wheelchairs are tested by subjecting the wheelchair to a frontal impact at 30 mph (48 km/h). A wheelchair is loaded with a suitable crash test dummy, accelerated to 30 mph and brought to a controlled stop, simulating a frontal impact generating 20 g of force on the crash test dummy. Permobil has crash tested the TiLite X with Transit Tie-Down Option in one configuration and the wheelchair tested was compliant with ISO 7176-19:2022. The wheelchair was tested utilizing a 170 lb (77 kg) crash test dummy, which corresponds to an occupant weight of 165 to 300 lbs (75 to 136 kg), and used a vehicle-anchored shoulder belt and a wheelchair-anchored pelvic belt. To minimize risk when travelling in motor vehicles, make sure that any adjustable accessories are configured as close as possible to the crash tested chair configuration. The wheelchair tested had a total mass of 26 lbs (12 kgs) was configured as follows:


Seat Width	16.0 in. (40 cm)	Front Angle	70°	Push Handles	None	Rear Wheels	24" (540) Wheel
Seat Depth	16.0 in. (40 cm)	Seat to Footrest	15 in. (38 cm)	Center of Gravity	2.0 in. (5 cm)	Tires	Pneumatic
Front Seat Height	17.5 in. (44.5 cm)	Foot Plates	Composite Flip Up	Camber	0°	Anti-Tips	Standard
Rear Seat Height	15.5 in. (39.5 cm)	Back Height	20 in. (51 cm)	Caster Wheels	5" Soft Roll		

- This option is labeled "Transit Tie-Down Option" on the TiLite X order form. When ordered, product ships with four factory-installed, transit tie-down brackets and two factory-installed, wheelchair-anchored, pelvic belt brackets. The four tie-down brackets are used to secure the wheelchair within the vehicle using a four-point, strap-type wheelchair tie-down system. The two pelvic belt brackets provide anchorage points to secure a crashworthy wheelchair-anchored pelvic belt that conforms to the requirements of ISO 7176-19:2022 and that can be used in conjunction with a vehicle-anchored shoulder belt with a standard lower-anchorage connector for effective crashworthy three-point belt restraint in a motor vehicle. The occupant is to be secured within the wheelchair using a three-point restraint system consisting of a wheelchair-anchored pelvic belt and a vehicle-anchored shoulder belt. The TiLite X with Transit Tie-Down Option was tested using a Q'STRAIT Wheelchair-Anchored Pelvic Belt. However, the TiLite X with Transit Tie-Down Option does not ship with such a pelvic belt as standard equipment. The Q'STRAIT Wheelchair-Anchored Pelvic Belt may be ordered from TiLite as an optional accessory at the time of purchase or after. TiLite recommends that the TiLite X be used only with a vehicle-anchored shoulder belt and a wheelchair-anchored pelvic belt. While the Transit Tie-Down Option is compatible with, and can be purchased with, additional seat sizes, optional components and varied configurations, TiLite makes no claim that any other components or configurations have been tested beyond the above-described ISO 7176-19:2022 conforming configuration. TiLite does not claim that its Transit Tie-Down Option will prevent injury or death in the event of a motor vehicle accident.
- Annex D of ISO 7176-19:2022 provides a method of testing a wheelchair for its ability to accommodate vehicle-anchored pelvic and shoulder belts. When tested in accordance with Annex D of ISO 7176-19:2022, the TiLite X had a score of "Excellent" for ease of proper belt positioning and an overall score of "Excellent" (16 of 16 points). The ease of access to, and maneuverability in, motor vehicles can be significantly affected by wheelchair size and turning radius. Smaller wheelchairs and/or wheelchairs with a shorter turning radius will generally provide greater ease of vehicle access and maneuverability to a forward-facing position.

Transportation, continued


Securement in a Motor Vehicle

Make sure that the wheelchair is secured in a forward-facing orientation using all four attachment points prior to travelling in the motor vehicle. To secure the wheelchair, locate the four tiedown points, marked with the hook symbol shown below: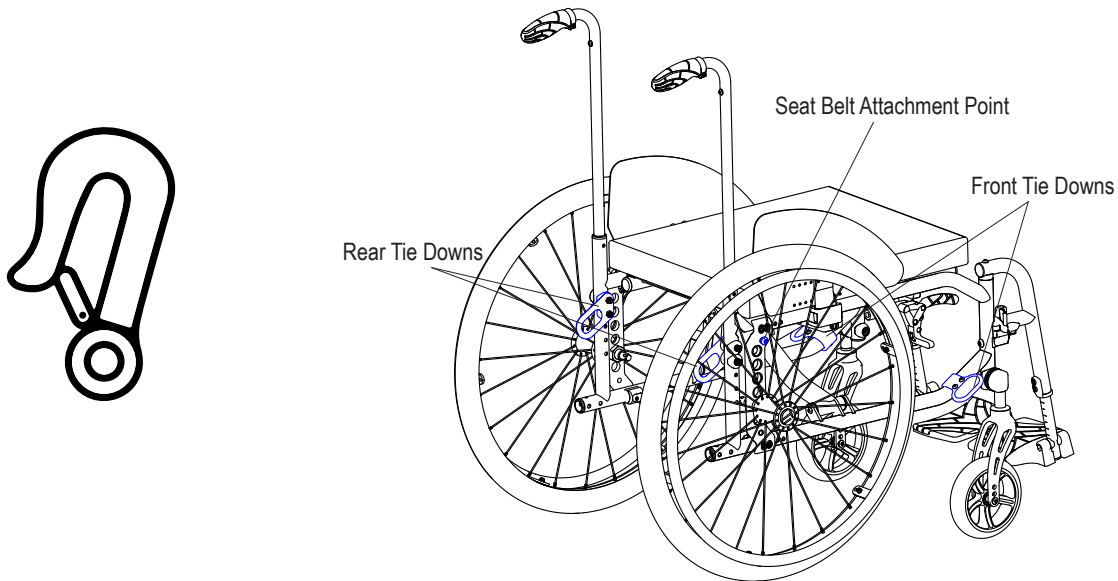


The tie downs on this wheelchair are compatible with four-point strap-type securement point end fittings.

Once the wheelchair has been secured, the occupant shall be secured with the vehicle mounted belt restraints. Ensure that the pelvic belt is worn low across the front of the pelvis, so that the angle of the pelvic-belt restraint is within the zone of 30° to 75° to the horizontal. If feasible, a steeper (greater) pelvic belt angle between 45° to 75° to the horizontal is preferred. Shoulder-belt restraints should fit over the middle of the shoulders and belt restraints should be adjusted as snugly as possible, consistent with occupant comfort. Belt webbing should not be twisted when in use.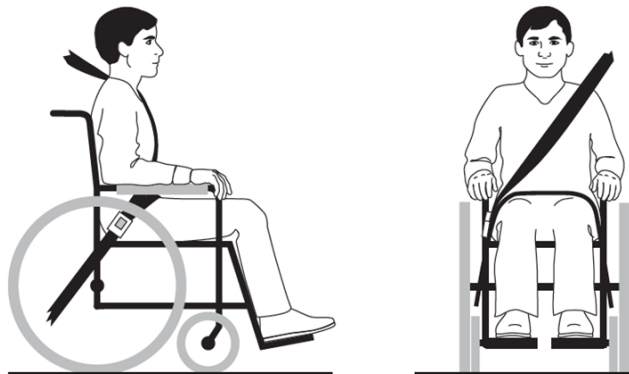



The belt restraints shall not be held away from the body by wheelchair components or parts, such as the wheelchair arm supports or wheels.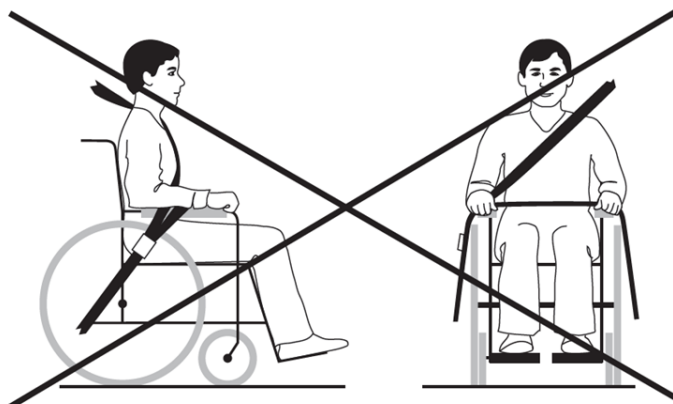

Product Specifications

Please Note:


- The serial number and other important information can be found on the label applied to the rear of the frame.
- To order replacement parts, contact the product provider or Customer Support.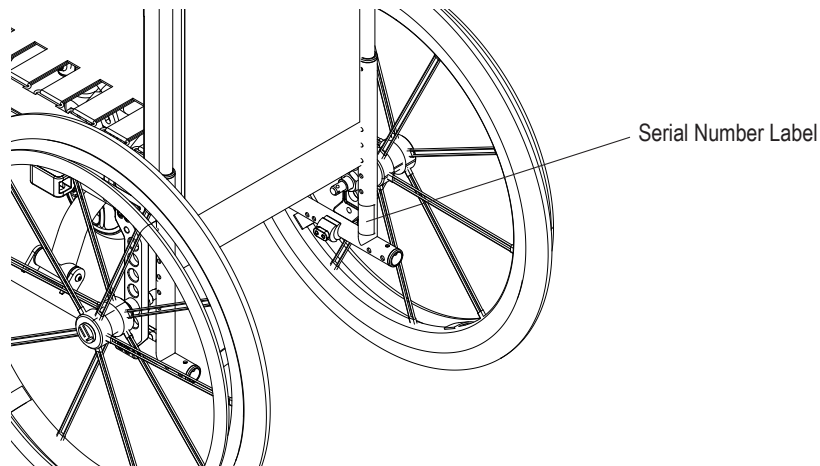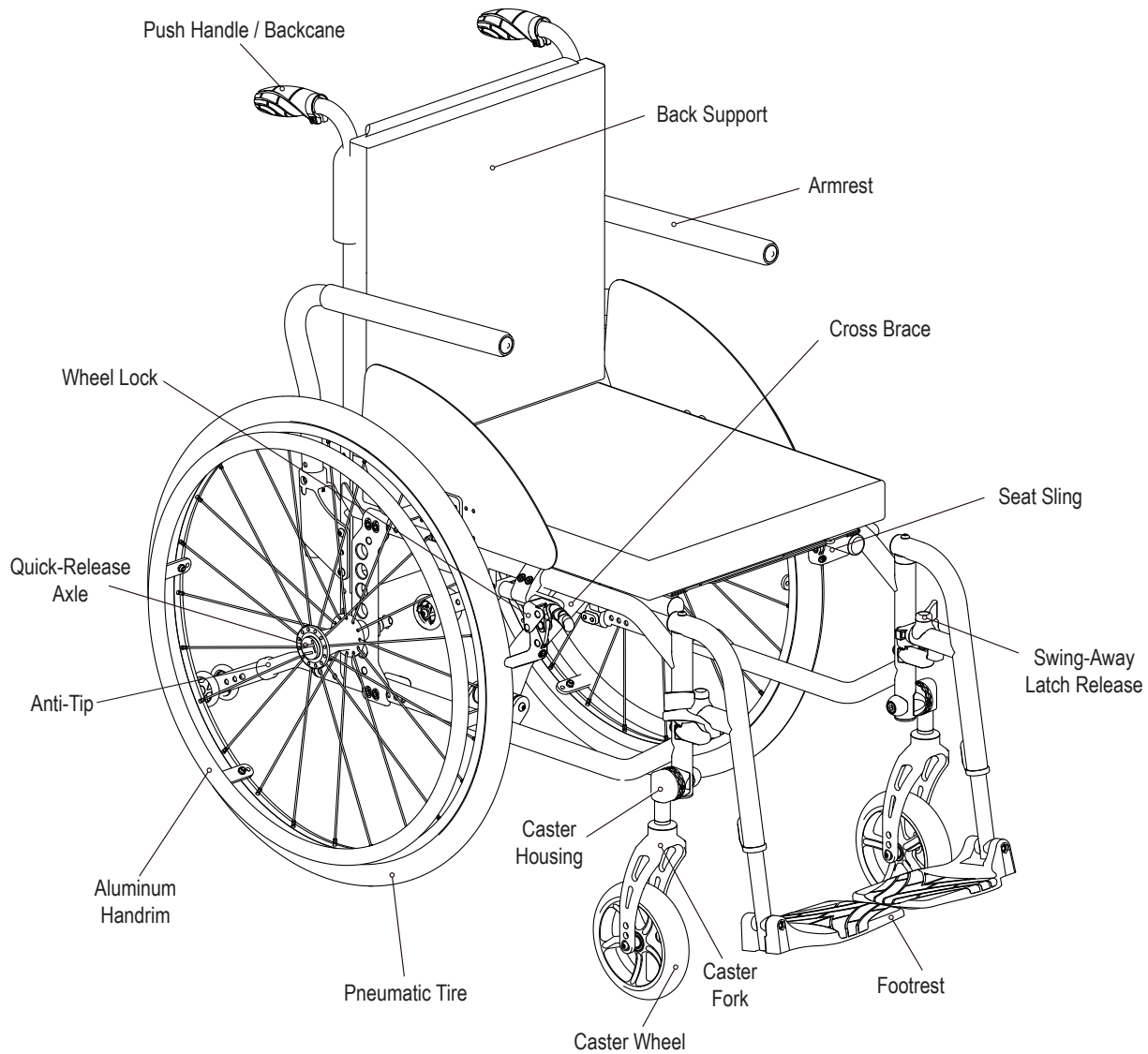

Operation Instructions

Folding the Wheelchair

1. To fold the wheelchair, start by removing the seat cushion and hangers.
2. Lift up on the Strap in the middle of the seat sling until the two halves of the wheelchair come together.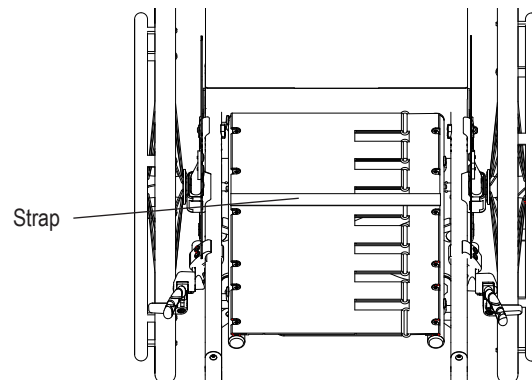

Quick Release Axles


⚠ Warning: Keep axles and receivers clean and away from dirt and grime, and avoid moisture. If they do get wet, dry the axle and apply a light coating of "dry" silicone or Teflon-based lubricant to prevent rust and provide easy insertion and removal.

1. To attach the wheel, fully press the quick release button (or actuate the lever on the lever release style of axle) and while holding down, insert the axle into the receiver.

Make sure the axle is fully inserted into the receiver and locked into place (quick release button or lever returns to original position) and the wheel does not come out when pulled.

If the button is not fully extending when inserted, the wheel is not locked into place and can fall out of the receiver causing bodily harm or injury.

2. To remove the wheel, push down completely on the quick release button and while it is fully pressed, pull the wheel assembly outwards and away from the receiver.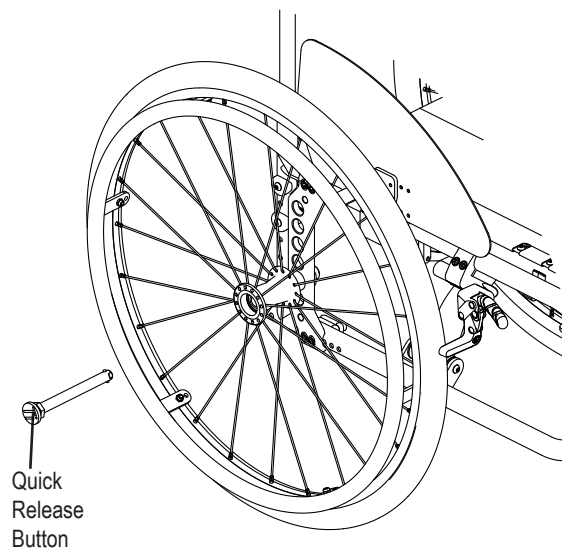

Operation Instructions, continued

Back Support


The wheelchair may be equipped with a fold down back support.

To fold down the back support:

1. Push down on the Latch Release Levers on both sides of the wheelchair.
2. Push the back support forward.

To set the back support up for use:


1. Pull back on the back support until the back support is completely up and the Latch Release Levers engage.
2. Make sure the back support is locked in the upright position by pressing the back support forward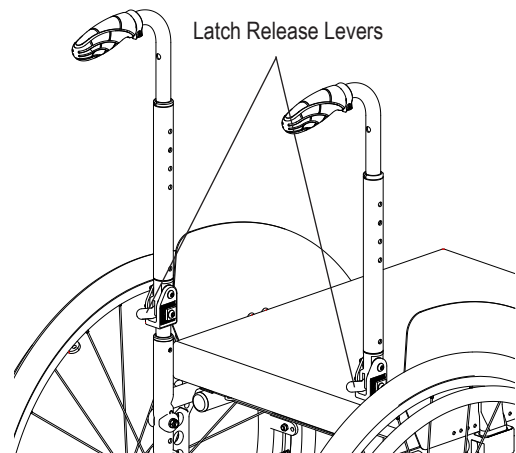

Upholstery


Seat Upholstery:

1. The upholstery is tension adjustable by hook and loop fastener straps. To adjust the tension on the seat upholstery, access the straps underneath the seat and adjust to preference. For optimal wheelchair performance and responsiveness, it is recommended to keep the seat sling tightly fastened to the wheelchair frame.
2. To tighten the sling, begin by transferring out of the wheelchair. Start with a strap towards the middle of the seat sling. Undo the hook and loop fastener strap, pull tightly around the attachment ring and re-secure on the exposed hook and loop fastener of the same strap.
3. Continue this same process with the strap behind the first strap. It is best to alternate with consecutive straps in front and back until there are no more straps in the back and then finish the remaining straps, working towards the front.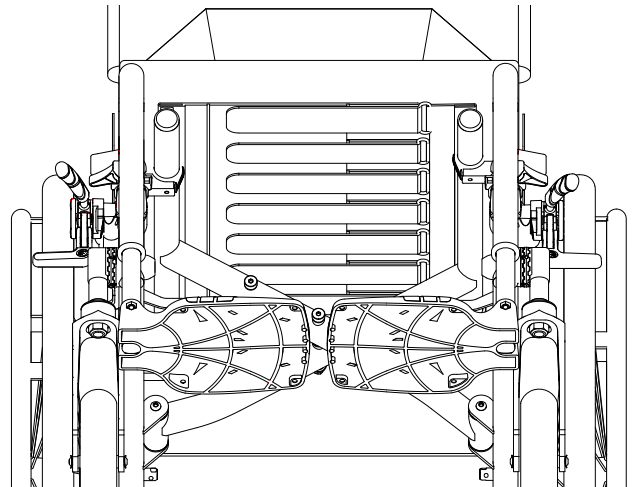

Back Upholstery:

1. To adjust the tension of the back upholstery, begin by moving away from contact with the back upholstery. Remove the ties connecting the grommets in the privacy flap to the rear of the chair frame. Pull the privacy flap up and over the top of the back support until it is completely removed from the wheelchair (image shown with back already removed).
2. Undo the hook and loop fastener straps and tighten or loosen to preference. Be careful not to over-tighten the very top straps as it is possible to pull the vertical back canes too close together which may affect the performance of the folding operation of the back support.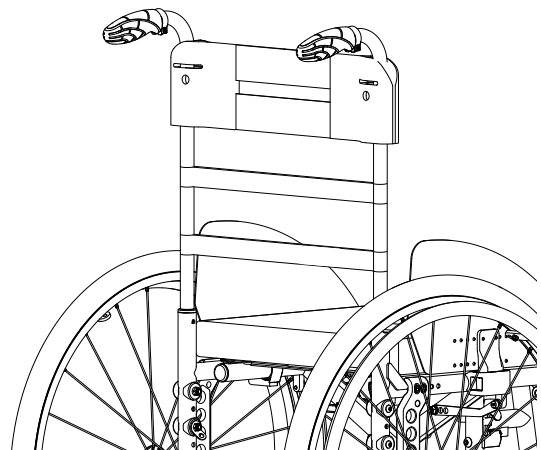


Operation Instructions, continued

Anti-Tips

To install the anti-tips:


1. Press down the exposed button on the end of the anti-tip tube and slide into the receiver at the bottom of the camber tube vertical strut.
2. Make sure the button of the anti-tip tube pops into the hole on the receiver and locks it into place.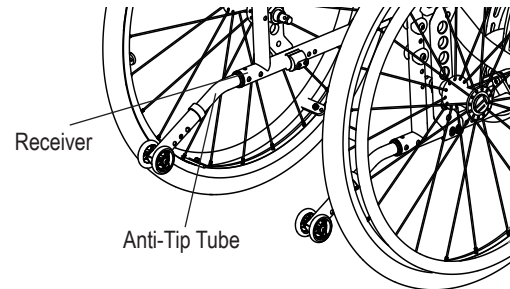

The wheelchair may be equipped with user friendly anti-tips.

To flip the anti-tips up:

1. Press down on the thumb button on the top of the anti tip rotation mechanism. With the button pressed down, lift up on the anti-tip, rotating it upwards.
2. Release the button and make sure the anti-tip latches in the upper position.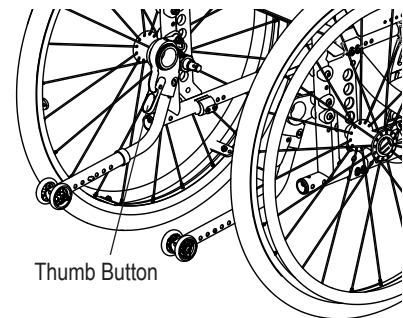

To engage the anti-tips up:

1. Press down on the thumb button on the top of the anti tip rotation mechanism. Press down on the anti tip, rotating it downwards.
2. Release the button and make sure the anti-tip latches in the downward position before using the wheelchair.


Wheel Locks

The wheel locks on the TiLite X come pre-installed but can be adjusted to an individual's preference.

To adjust:

1. Loosen the adjustment screws on the wheel lock clamp.
2. Slide the lock assembly forward (less lock tension) or backwards (more lock tension).

IMPORTANT! The effort required to engage or disengage the wheel lock is an indication of grip strength against the wheel and the wheel lock's performance. Also, a reduction in lock strength likely indicates an underinflated tire with less than optimal tire pressure.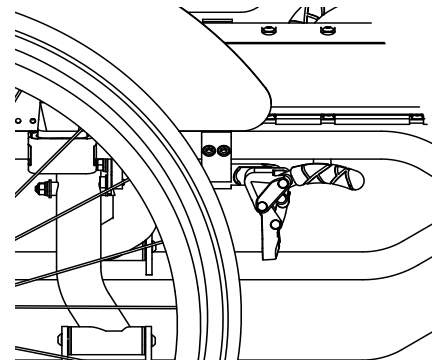


Inflating Tires


The wheelchair may be equipped with pneumatic rear tires.

To check tire pressure:

1. Remove the valve cap and use a tire pressure gage to measure the pressure inside the tire.
2. To determine the maximum pressure for the tire, read the sidewall of the tire. If the measured pressure is less than the tire's listed maximum pressure, use an air pump or electric compressor to inflate the tire to the maximum rated pressure.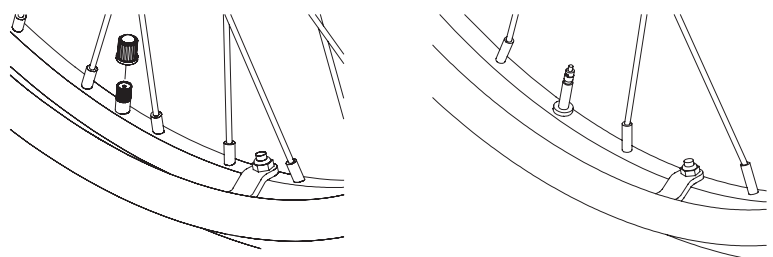

IMPORTANT! If the wheelchair is equipped with X-Core solid rear wheels, the maximum inflation pressure is 75 PSI (517.07 kpa), even if the tire is rated higher. Inflating the rear tires beyond 75 PSI (517.07 kpa) can cause permanent damage to the wheels and potential harm to the user.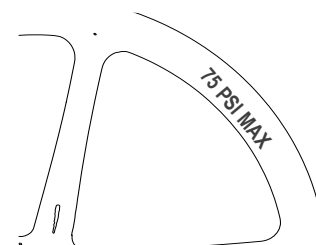

Operation Instructions, continued

Positioning Belt


The wheelchair is available with a black nylon, hook and loop fastener adjustable positioning belt. To adjust, undo the strap and set the tension as desired.

Side Guards

The wheelchair may be equipped with removeable side guards.

To remove: Lift the Side Guard up and out of the Receiver.

To install: Insert the Side Guard into the Receiver on the frame. Make sure that the Side Guard is fully seated in the receiver prior to using the wheelchair.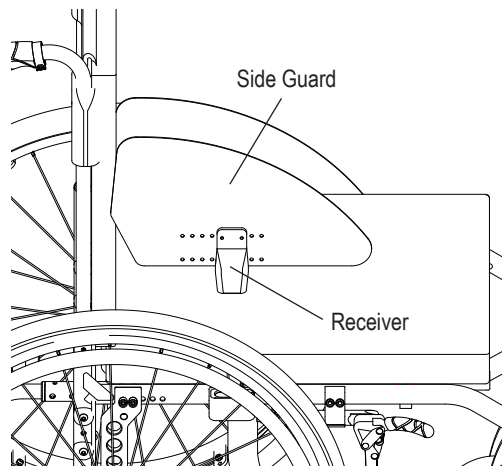


Armrests

Tubular Swing Away:

The wheelchair may be equipped with removeable tubular swing away armrests.

To swing away: Lift the armrest up until the tab dis-engages from the groove on the receiver, and swing the armrest outwards away from the wheelchair.

To remove: Lift the armrest all the way out of the receiver.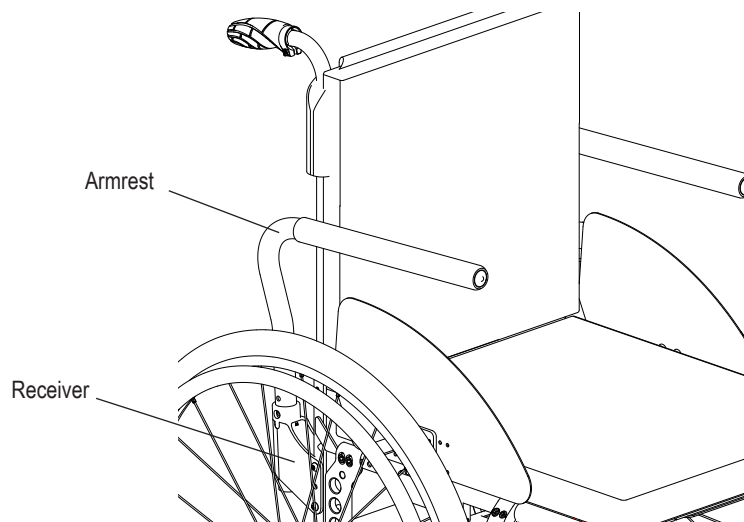

Operation Instructions, continued

Armrests, continued


Two-Point Back Flip T-Arm:

The wheelchair may be equipped with two-point flip T-armrests.

To flip the armrest back: Press the release lever within the side guard area of the armrest and rotate up and back until the armrest is near vertical.

To engage the armrest: Rotate the arm down until the armrest locks into the receiver on the frame and the latch engages. Make sure the arm is engaged by attempting to lift the armrest before using the wheelchair.


To adjust the height of the armrest: Pull the height adjustment plunger above the latch lever. Slide the armrest to the desired height and let go of the plunger. Make sure the plunger engages with a hole on the armrest and locks the adjustment prior to using the wheelchair.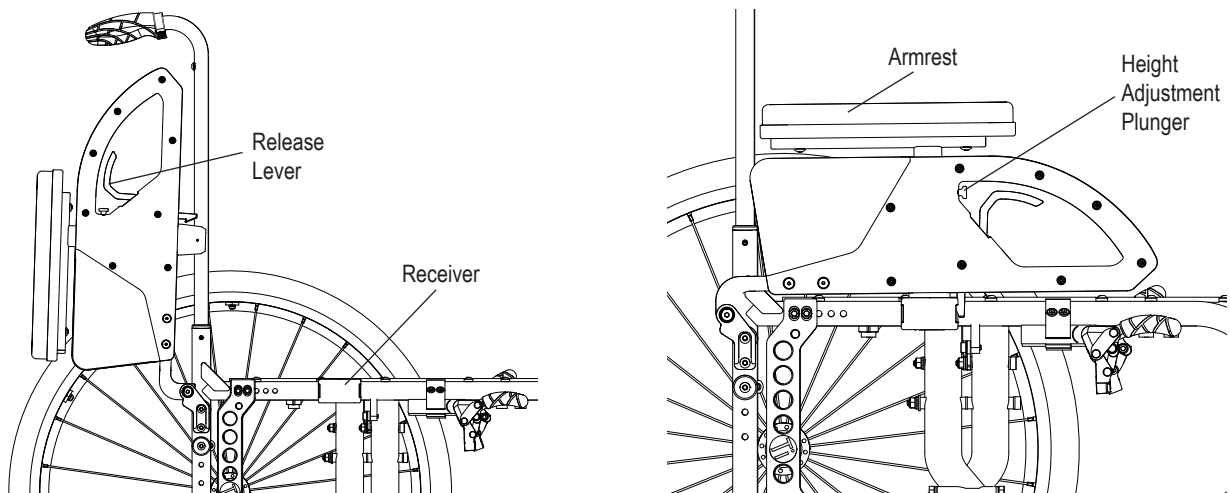

Removable T Arm:

The wheelchair may be equipped with removable T-armrests

To adjust the height of the armrest: Pull the height adjustment plunger above the latch lever. Slide the T-arm to the desired height and let go of the plunger. Make sure the plunger engages with a hole on the T-arm and locks the adjustment prior to using the wheelchair.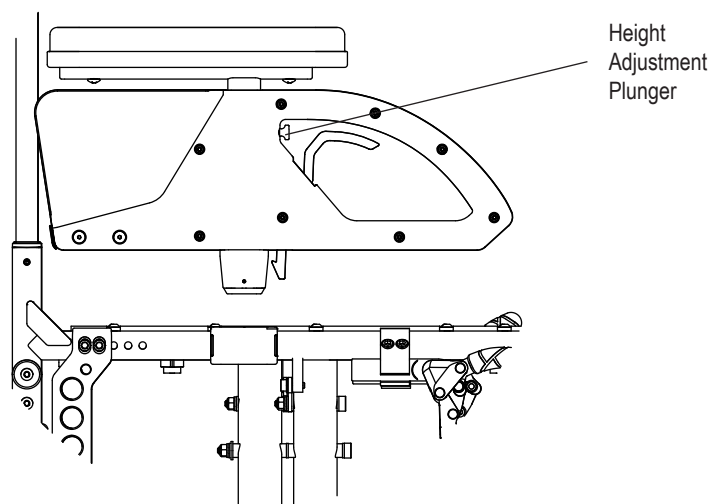


Operation Instructions, continued

Integrated Residual Limb Support

The wheelchair may be equipped with an integrated residual limb support.

To swing the residual limb support away: Push the lever on the release mechanism off to the side as this will disengage the latch. When the latch is open, push the residual limb support outwards to swing it away. Once the residual limb support is rotated sufficiently outwards, it can be removed from the wheelchair by lifting upwards.

To install the residual limb support: Start with the hanger in the swung away position. Insert the residual limb support into the front tube and ensure the residual limb support is fully seated. Once seated, swing the support towards the middle of the wheelchair. The latch should engage automatically. Make sure the latch is engaged before using the residual limb support.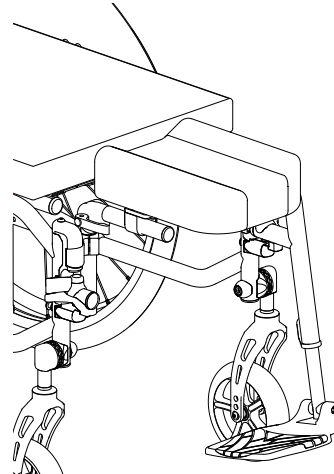

Elevating Leg Rests


The wheelchair may be equipped with elevating leg rests (ELR's).

To swing the ELR outward: Push the release lever on the release mechanism and rotate the ELR outward.

To remove the ELR from the wheelchair: Push the lever on the hanger release mechanism and swing the ELR outward. Remove the ELR by lifting away from the frame.

To install the ELR:

1. Align the top pivot with the frame hole. Holding the ELR, not the release lever, swing the ELR until the hanger release mechanism locks into place.
2. Fold down the footplate and rotate the calf pad until it comes to a stop for placement under the calf.
3. Press the ELR angle adjustment lever and raise the ELR gradually until the desired leg height is reached. If a lower ELR angle is desired, push the ELR angle adjustment lever gradually, holding on to the ELR while lowering the ELR into position.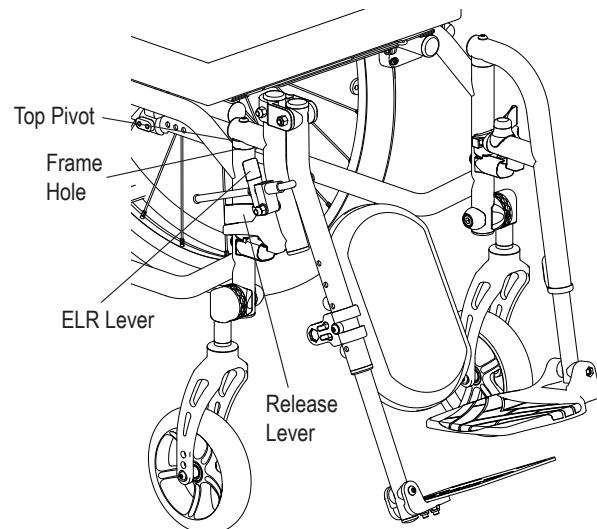

Removing and Installing the Swing Away Hangers


The wheelchair is equipped with swing away hangers.

Removing the Swing Away Hangers:

1. To swing the hanger away, push the lever on the hanger release mechanism off to the side. This will disengage the latch.
2. Once the latch is open, push the hanger outwards to swing it away. When the hanger is rotated sufficiently outwards, it can be removed from the wheelchair by lifting upwards.

Installing the Swing Away Hangers:

1. To install the hanger, start with the hanger in the swung away position.
2. Insert the hanger into the front tube and ensure the hanger is fully seated.
3. Once seated, swing the hanger towards the middle of the wheelchair. The latch should engage automatically. Make sure the latch is engaged before using the hangers.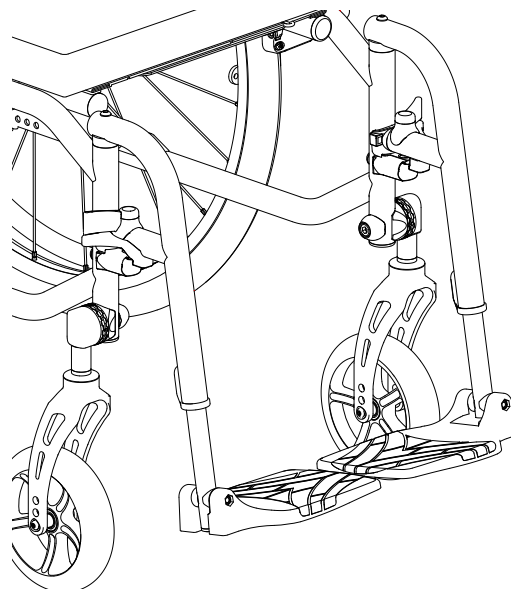

Cleaning and Disinfecting

 **Caution:** Clean the product regularly and when it becomes soiled.

Recommendation: When washing by hand, use room-temperature water.

Wheelchair Frame and Accessories

 **Cautions:**

- DO NOT wash the wheelchair with a pressurized stream of water.
- DO NOT expose the wheelchair to corrosive agents such as acids, alkalis, or salts or cleaners. Corrosive materials may damage the wheelchair components and compromise its safety and performance.


To Clean: Wipe clean using a damp cloth and mild soap. Wipe until all visible soil has been removed from the wheelchair. Rinse with clean water. Wipe dry with a clean cloth and then allow to air dry completely.

Sling Seat and Back Upholstery

 **Caution:** DO NOT machine wash or machine dry the sling seat or back upholstery.

To Clean: Regularly hand wash the sling seat and back upholstery using water and a mild detergent. While cleaning, carefully check for sagging, rips or tears and replace if necessary. Allow to air dry completely before using the wheelchair.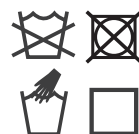

IMPORTANT! After the sling seat is dry, check the tension while seated in the wheelchair to make sure the sling does not touch the crosstube beneath the sling seat.

Care Symbols

Wipe with a damp cloth

Hand wash

Air dry

Do not machine wash

Do not tumble dry

Troubleshooting

For additional assistance, contact the equipment provider, distributor, or Customer Support.

Wheelchair drifts to one side	• Check that all wheels can spin freely and are not difficult to rotate. Repeat for the caster forks. • On a level surface, check that all four wheels touch at the same time. If one of the caster wheels is floating, take the wheelchair to a qualified equipment provider for adjustment.
Casters have difficulty rolling	• Make sure the caster axle area between the inside face of the fork and the caster is free from any blockages or foreign objects which can get wound around the axle. Remove any blockages or foreign objects, like thread or hair, that may be found. The easiest way to do this is to remove the caster from the fork. • Check the condition of the caster bearings to make sure they are spinning freely and are free from dirt and rust. • Maintenance intervals can be extended by avoiding rolling through standing water. If casters get wet, dry them off and apply a light coating of "dry" silicone or Teflon-based lubricant. Routinely perform this maintenance to continue the optimal performance of the wheelchair.
Rear wheels aren't rolling properly	• Check the tire pressure. The wheelchair will perform the best when the tires are at their optimal level of inflation. If tire pressure is low, add air. • If a wheel still feels sluggish, check that the rear axles are free and clear of any contaminants and check the wheel bearings to make sure they are spinning freely and are free from dirt and rust. Just like the caster bearings, if exposed to moisture, dry them off and apply a light coating of "dry" silicone or Teflon-based lubricant. • Check the alignment of the wheels to make sure that the wheels are not rubbing against the wheelchair or side guards. This can be an indication of incorrect rear wheel spacing.
Wheel locks don't engage properly	• Check that the tires are inflated to the proper pressure. If tire pressure is low, fill to the max recommended pressure setting on the tire sidewall. • The wheel lock may have slid inside the wheel lock clamp. If so, loosen the adjustment screw and slide the lock forward or backwards for the desired tension, and retighten the screw to the recommended torque setting. • Check that the lock is perpendicular to the tire and hasn't rotated in the clamp, preventing full engagement with the tire.
Seat sling tension is not adequate	• If the seat sling tension needs to be adjusted, follow the instructions in the Operation Instructions section.
Push handles are loose	• Push handles should be securely fastened to the back vertical tubes. If any movement in the handle becomes noticeable, please check that the attachment hardware is properly tightened.
Side guard movement	• Unusual sounds or excessive movement in the side guards can be an indicator of loose hardware. Check and tighten any loose hardware.
Footrests are loose	• Footrest extensions should be securely fastened to the hangers. If any movement in the footrest extension becomes noticeable, please check that the attachment hardware is properly tightened.

Maintenance

Reference the table below for preventative maintenance information. Please consult a qualified wheelchair provider if not confident or uncomfortable performing any of the tasks listed below. For additional information regarding service, please reference the service manual for the wheelchair found on the Permobil website.

Inspection Item	Initially	Weekly	Every 3 Months
General - Make sure the wheelchair rolls straight (does not drag or pull to one side) - Inspect for loose, missing, broken, worn, and/or damaged hardware	X X	X X	
Rear Wheels - Inspect for cracked, bent or broken spokes - Make sure all spokes are uniformly tight - Inspect hubs and rims for cracks and wear - Inspect for "trueness" (excessive side movement when wheel lifted and spun indicates lack of "trueness")	X X X X	X X	 X X
Quick-Release/Quad-Release Axles - Make sure axles are free of dirt and lint - Wipe axle with cloth using either a "dry" silicone spray or a "dry" Teflon-based lubricant	X X		X X
Rear Tires - Check for proper inflation as per sidewall of pneumatic tire - Inspect for cracks, wear and flat spots	X X	X X	
Camber Tube / Axle Sleeves - Make sure camber clamps are securely tightened - Make sure camber plugs are secure in camber plug clamps and axle sleeves are secure in camber plugs	X X		X X
Handrims - Inspect for signs of rough edges or peeling - Make sure handrims are securely attached to the rim	X X		X X
Wheel Locks - Make sure wheel locks embed properly into tire and prevent the wheelchair from moving when engaged - Make sure wheel locks do not interfere with tire when disengaged - Make sure pivot points are not worn or loose	X X X	X X X	
Seat Sling and Back Upholstery - Inspect for rips or sagging - Inspect fastening flaps to make sure they are securely attached - Inspect while seated to make sure the sling supports the user's weight and does not "bottom out" on the wheelchair frame cross tube - Make sure seat upholstery bolts are securely tightened.	X X X X	X X	 X X
Casters/Forks - Inspect wheels and tires for cracks and wear - Inspect wheel/fork assembly for proper tension by spinning caster (caster should come to a gradual stop); if caster wobbles, tighten the axle - Inspect wheel/fork assembly/stem bolt to make sure that stem bolt is secure (wheel/fork assembly should not have excessive play relative to the stem bolt, but should rotate freely. A loose nylock nut may cause the fork to wobble or detach and result in injury or equipment damage); if necessary, tighten stem bolt - Make sure that wheel bearings are clean and free of moisture and debris	X X X X	 X	 X X
Footrests / Hangers Inspect to make sure all hardware is securely attached	X	X	
Anti-Tips - Inspect anti-tip wheel for cracks and wear - Inspect to make sure the anti-tip receptacle is securely attached	X X	 X	X
Frame Inspect for cracks	X		X

Storage, Disposal and Recycling

Storage: Clean the product. Cover the entire wheelchair, including casters, rear wheels and their interacting surfaces, to protect the wheelchair from moisture, contaminants, pests and damage. After storing, inspect the product to make sure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of a product's useful life (5 years), treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incineration must be performed by a qualified, licensed waste management facility.

Recycling: Contact a local recycling agency to determine recycling options for the product.

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