**permobil** | SEATING + POSITIONING

## ROHO® Hybrid Select Cushion

Operation Manual

s h a p e f i t t i n g t e c h n o l o g y®

## Contact Information

 ROHO, Inc.  
1501 South 74th St.  
Belleville, IL 62223 U.S.A.  
U.S.A.:  
800-851-3449  
Fax 888-551-3449  
Customer Support:  
orders.roho@permobil.com  
Outside the U.S.A.:  
618-277-9150  
Fax 618-277-6518  
intlorders.roho@permobil.com  
permobilus.com

### Warning

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

### Caution

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

## Incident Reporting

In the event of a serious incident or injury, contact Customer Support.

## Contents

Intended Use	2
Important Safety Information	
Parts Detail	3
ISOFLO Memory Control	
Offloading Option	
Setup	4-5
Cover and Cushion Components	6
Cleaning and Disinfecting	7
Troubleshooting	8
Product Specifications	
Storage, Transport, Disposal and Recycling	
Limited Warranty	
Español	9
Français	17
Português	25

## Symbols Glossary

 Manufacturer	 Fits	 Serial number	 Front of product	 Cleaning
 Date of manufacture	 Model number	 Weight limit	 Back of product	 Disinfecting
 Consult instructions for use	 Quantity	 Size (width x depth x height)	 Bottom of product	 Meets flammability
 Batch code	 Medical device			

## Please Note

Supplier: This manual must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Products may be covered by one or more U.S. and foreign patents and trademarks, including DRY FLOATATION®, ROHO® and shape fitting technology®. Zytel® is a registered trademark of DuPont.

Product information is changed as needed; current product information is available at permobilus.com. Contact Customer Support for previous versions of product information.

ROHO, Inc. is part of Permobil Seating and Positioning.

© 2020, 2021 Permobil

Rev. 2021-02-23

## Intended Use Statement

The ROHO Hybrid Select Cushion (Hybrid Select) is an adjustable, air-filled, wheelchair support surface combining a ROHO DRY FLOTATION cellular design cushion insert with a contoured foam base, and is intended to conform to an individual's seated shape in the pelvic loading area to provide skin/soft tissue protection, positioning, and an environment to facilitate wound healing. Optionally, through removal of the IT Air Pad insert, the Hybrid Select is also intended for offloading of the ITs. There is a weight limit of 500 lbs. (227 kg), and the cushion must be properly sized to the individual. The Hybrid Select must be used with the supplied cover.

*A clinician who is experienced in seating and positioning should determine whether the product is appropriate for the individual's particular seating needs. The product's intended use statement alone should not be relied upon to make that determination.*

*The medical products featured in this document are intended to be only one part of an overall care regimen that includes all seating and mobility equipment and therapeutic interventions. A clinician who is experienced in seating and positioning should identify this care regimen after evaluating the individual's physical needs and overall medical condition.*

*A clinician should also evaluate for visual, reading, and cognitive impairment to determine the need for caregiver assistance or the need for other assistive technology, such as large-print instructions, to ensure proper product use.*

## Important Safety Information

### Warnings:

- Skin/soft tissue breakdown can occur due to a number of factors, which vary by individual. Check skin frequently, at least once a day. Redness, bruising, or darker areas (when compared to normal skin) may indicate superficial or deep tissue injury and should be addressed. If there is any discoloration to skin/soft tissue, STOP USE immediately. If the discoloration does not disappear within 30 minutes after disuse, immediately consult a healthcare professional.
- Check inflation frequently, at least once a day.
- DO NOT use a product that is underinflated or overinflated, because 1) the product benefits will be reduced or eliminated, resulting in an increased risk to skin and other soft tissue, and 2) the individual may become unstable and vulnerable to falling. Carefully follow all "Setup" instructions. If the product does not appear to be holding air, or if you are not able to inflate or deflate the product, see "Troubleshooting". Immediately contact your equipment provider, distributor, or Customer Support if the problem persists.
- DO NOT use the components of this product separately. Assemble and use this product only as instructed in this manual.
- The cushions, foam and cover MUST be compatible sizes and MUST be used as directed in this manual. IF THEY ARE NOT: 1) The benefits provided by the product may be reduced or eliminated, increasing the risk to skin or to other soft tissue, and 2) the individual may become unstable and vulnerable to falling.
- DO NOT use the cushion on top of, or in conjunction with, any other products or materials, except as indicated in this manual. Using non-compatible products or materials, whether inside or outside the cover: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.
- DO NOT place obstructions between the individual and the cushion. The cushion and the cover MUST be compatible sizes and MUST be used as directed in this manual. Except for the compatible cover provided with this product, placement of any items between the individual and the cushion: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.

### Warnings:

- DO NOT use any part of the product as a water flotation device (e.g. a Life Preserver). It will NOT support you in water.
- The air cell surfaces will acclimate to the surrounding temperature. Take appropriate precautions, especially when the air cells will contact unprotected skin.
- DO NOT expose the product to high heat, open flames, or hot ashes. Testing or certification claims, including for flammability, may no longer apply to this device when it is combined with other products or materials. Check testing and certification claims for all products used in combination with this device.
- If you are unable to perform any tasks described in this manual, seek assistance by contacting your clinician, equipment provider, distributor, or Customer Support.

### Cautions:


- Do not use a pump, cover or repair kit other than those compatible with the product.
- DO NOT modify the cushion or any components. Doing so may result in damage to the product and will void the warranty.
- DO NOT allow the air cells to come into contact with oil-based lotions or lanolin, which may degrade the material.
- Prolonged exposure to ozone may degrade materials used in the cushion, affect the performance of the cushion, and void the product warranty.
- Changes in altitude may require adjustment to the cushion.
- Keep the cushion away from sharp objects.
- Air cells that have been in temperatures lower than 32° F (0° C) may exhibit unusual stiffness and should be warmed to room temperature.
- Periodically, check the cover and components for wear and damage and replace if necessary.
- Carry the product by the cushion base or by the carrying handle on the cover. Use of any other components to carry or pull the cushion may damage the product.

## Parts Detail

**Package contents:** Hybrid Select Cushion, Hybrid Select Cover, hand pump, repair kit, operation manual, limited warranty supplement, product registration card.


*Note: To order replacement parts, contact your product provider or Customer Support.*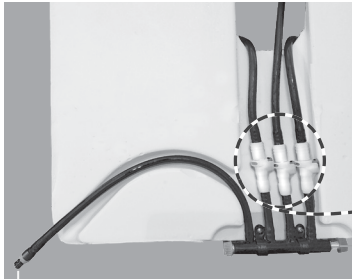

Inflation Valve

Quick Disconnect (x 3)


ISOFL0 Memory Control

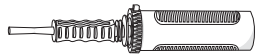
Hand Pump

Repair Kit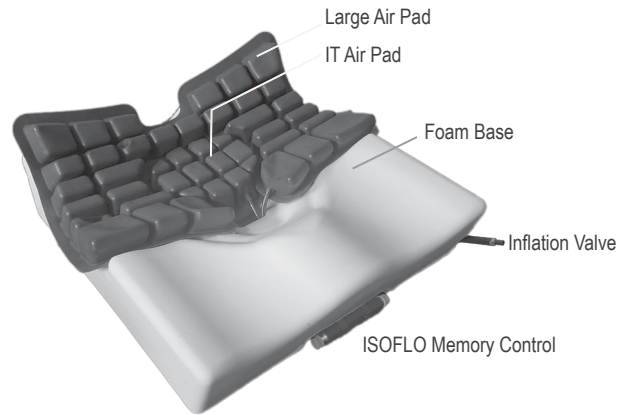


## ISOFL0 Memory Control

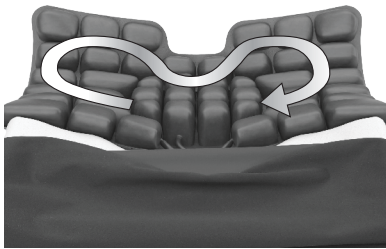**Unlock = GREEN KNOB**  
 Unlocks the air flow.  
 Allows the air to flow freely through all of the air cells.

**Lock = RED KNOB**  
 Keeps the airflow locked within the IT Air Pad (if using) and the right and left sides of the Large Air Pad.

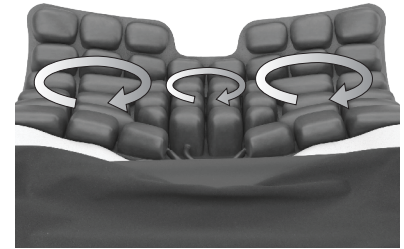## Offloading Option

**⚠ IMPORTANT!** If you are using the Hybrid Select without the IT Air Pad (as an offloading cushion) follow these instructions, and then continue with the Cushion Setup instructions. Consult with a clinician to adjust the cushion as prescribed.

1. Remove the cover. See the "Cover and Cushion Components" section in this manual.
2. Locate the tubing that attaches the IT Air Pad to the ISOFL0, and detach the quick disconnect.
3. Pull the tubing through the opening in the foam base and remove the IT Air Pad.

4. Make sure the Large Air Pad is positioned correctly on the foam base, with the hook/loop strips carefully aligned.

5. Store the IT Air Pad in a container that will protect it from moisture, contaminants, wear and damage. See additional storage instructions in this manual.

## Setup

**STOP HERE if using the Hybrid Select without the IT Air Pad. Refer to the "Offloading Option" in this manual before completing the Setup instructions on this page.**

**⚠ Warnings - Before Use:** Read all safety information and perform all steps below. Transfer to the cushion only after performing all steps included in "Setup" (two pages). Make sure the cushion is properly sized to the individual.

### SETUP - PART ONE: Remove the cover and unlock the ISOFLO.

1. Using a stable surface, such as a table, unzip and open the cover. Turn the product over and make sure the quick disconnects are securely connected to the ISOFLO.


*Note: When attaching the quick disconnects: The tubes can be attached in any order.*

2. Make sure the ISOFLO is unlocked. See the "ISOFLO Memory Control" section.

*Note: When the ISOFLO is unlocked, the air flows freely through all of the air cells. You will not be able to inflate the cushion unless the ISOFLO is unlocked.*

Inflation valve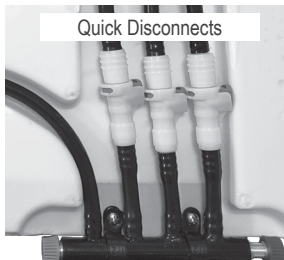

ISO FLO Memory Control

Unlock GREEN KNOB

### SETUP - PART TWO: Perform atmospheric inflation.

**⚠ IMPORTANT!** Make sure there is NO WEIGHT on the air pads. If the individual is sitting on the cushion, transfer off.

1. Open the inflation valve.

*Notes: To open, turn at least one full rotation. The inflation valve will spin freely when open. You can use the hand pump nozzle to open the inflation valve.*

2. Inflate with the hand pump until all of the air cells feel firm.

*Note: All air cells will inflate at the same time.*

3. Remove the hand pump, and DO NOT close the inflation valve.

4. Allow air to escape from the inflation valve. Listen (or place a hand in front of the inflation valve) to see if the airflow has stopped.


5. Close the inflation valve completely. The cushion is now inflated to atmospheric pressure.

### SETUP - PART THREE: Cover and place in the wheelchair.

1. Before closing the cover:

- Make sure the Large Air Pad is positioned correctly, with the hook and loop strips aligned.
- Make sure the Large Air Pad is securely attached to the foam base.
- Make sure there is NO GAP between the Large Air Pad and the foam base.

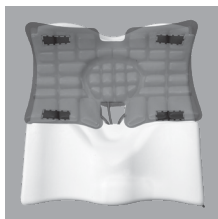
No gap!

2. Zip the cover closed (see cover installation instructions).

3. Place the Hybrid Select on the wheelchair (air cells facing up and non-skid surface of the cover facing down).


*Note: Use the symbol on the cover to help orient the cushion.*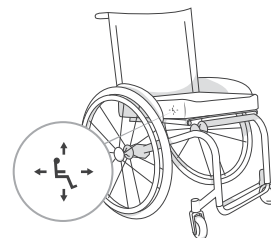

**Setup, continued**

**SETUP - PART FOUR: Transferring to the wheelchair (and POSITIONING, if needed):**

Before transferring, make sure the inflation valve is closed. The individual should transfer to the wheelchair and sit in their usual sitting position.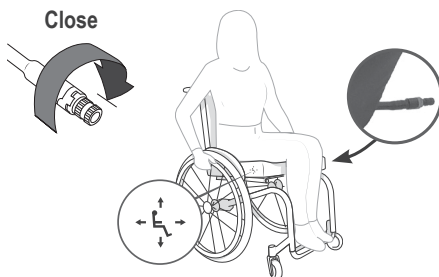

**If a clinician determines that POSITIONING is needed:** After inflating all air cells to atmospheric pressure and transferring to the wheelchair: Make sure the ISOFLO is unlocked. Perform the prescribed positioning. Then, lock the ISOFLO to maintain the position.

See the ISOFLO Memory Control section in this manual.

*Note: The inflation valve should be in the front, on the left side of the seated individual.*

**SETUP - PART FIVE: Transition and Immersion Checks**


**⚠ Warning:** Skin/soft tissue injury may result if the transition from air cells to the foam base is not level, due to overinflated or underinflated air cells. The air cells must not be overinflated. Refer to the warning about overinflation/underinflation in the "Important Safety Information" section of this manual.

**⚠ IMPORTANT!** It may be necessary to re-check and re-adjust the Transition and Immersion areas, to ensure proper inflation.

**Reminder! The ISOFLO must be open while adding or removing air.**

**FIRST: Transition Check: Make sure the transition from the air cells to the foam is level, in the area indicated.**


**Instructions:** Place a hand beneath the individual, where the air cells meet the foam base (indicated by the oval shape), and feel the transition from air pad to foam. Follow the instructions below.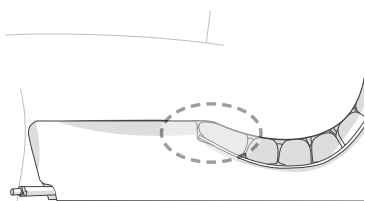

**Proper Inflation**

It feels LEVEL where the air cells meet the foam. There is no bump or ridge of overlapping air cells and foam.


**What to do:** Perform an immersion check, as shown below.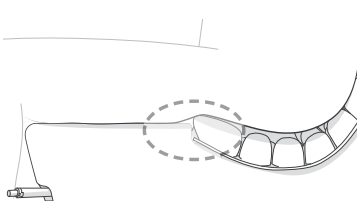

**Too much air in the air cells**

It feels like a bump or ridge of air cells lying on top of foam. The air cells are too high.

**What to do:** Remove a small amount of air and check again. Repeat as needed until the air cells are properly inflated. Then, perform an immersion check, as shown below.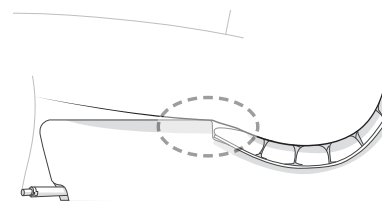

**Not enough air in the air cells**


It feels like a bump or ridge of foam. The air cells are not level with the foam.

**What to do:** Add a small amount of air and check again. Repeat as needed until the air cells are properly inflated. Then, perform an immersion check, as shown below.


**SECOND: Immersion Check: After the air cells are properly inflated to ensure a level Transition from air cells to foam, check to make sure there is enough air (or space) under the ITs.**

**With the IT Air Pad:** Place a hand beneath the lowest bony prominences (ITs) and make sure there is at least 1/2" of air. Add or remove a small amount of air, if needed. Repeat until the air cells are properly inflated.

**Without the IT Air Pad (Offloading Option):** Place a hand beneath the lowest bony prominences (ITs) and make sure that the ITs do not contact the foam. Add or remove a small amount of air, if needed. Repeat until the air cells are properly inflated.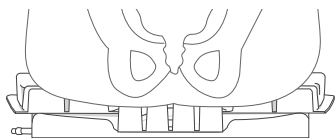

**Proper Inflation**  
(with the IT Air Pad)

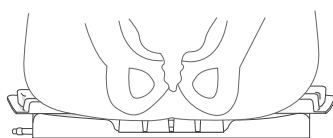**Underinflated**  
(with the IT Air Pad)

**Proper Inflation**  
(without the IT Air Pad)

**Underinflated**  
(without the IT Air Pad)

**⚠ WARNING: To ensure the product performs as intended:** Make sure that the back corners of the cushion contact the back canes of the wheelchair. Check for interference with other wheelchair components. Make sure that the wheelchair foot supports and back support are correctly positioned.

## Cover and Cushion Components

### To remove the cover:

Unzip the cover. Make sure the inflation valve is inside the cover. Push the ISOFLO into the slot and inside the cover. Place the product, air cells facing up, on a stable surface. Remove the cover.

### To remove the cushion components:

1. Detach all quick disconnects from the ISOFLO.
2. Pull the tubing through the opening in the foam base.
3. Remove the cushion components, releasing them from the hook/loop fastener strips.

### To install the cushion components:

1. If using the IT Air Pad, place it into the cutout area of the foam base. The tubing should face the opening in the foam base.
2. Place the Large Air Pad onto the foam base, carefully matching the hook/loop strips. Make sure the Large Air Pad is securely attached to the foam base. Make sure there is NO GAP between the Large Air Pad and the foam base.
3. Insert the tubing through the opening in the foam base.
4. Using the quick disconnects, attach each tube to the ISOFLO.


**⚠ IMPORTANT!** The flange of the IT Air Pad (if using) should lie under the flange of the Large Air Pad.

**When attaching the quick disconnects: The tubes can be attached in any order.**

5. Position the inflation valve tubing in the channel.



The "flange" is indicated by the dotted line.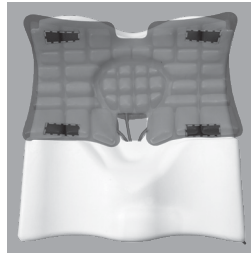

### To install the cover:

1. Confirm that the cushion components are installed correctly and connected to the ISOFLO.
2. Place the foam base on a stable surface. Slide the cover over the front of the cushion, and make sure it is snug.
3. Pull the ISOFLO out through the front slot in the cover. Push the inflation valve through the hole in the corner of the cover. Make sure the tubing is in the channel.
4. Place the cover around the rear corners of the foam base. Close the zipper.

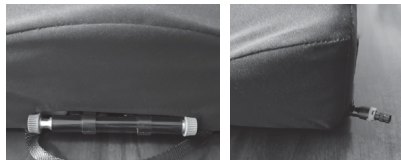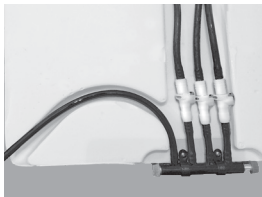**⚠ IMPORTANT!** Check to make sure the cover fits correctly on the contours of the cushion.

*Note: If used, hook fastener strips that were removed prior to washing should be replaced.*

## Cleaning and Disinfecting

### ⚠ Warnings:

- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. If the product becomes soiled, or before use by a different individual: clean, disinfect, and check the product for proper functioning.
- Clean the product regularly and when it becomes soiled.
- Follow all manufacturer bleach container and germicidal disinfectant safety guidelines.
- Note about sterilization: High temperatures accelerate aging and will damage the product assembly. The products featured in this manual are not packaged in a sterile condition, nor are they intended or required to be sterilized prior to use. If institution protocol demands sterilization: First, follow the cleaning and disinfecting instructions; then, open the inflation valve and use the lowest possible sterilizing temperature, but not to exceed 70°C (158°F), for the shortest time possible. DO NOT autoclave.

⚠ **IMPORTANT!** After cleaning and disinfecting and before use, follow all instructions to set up the cushion.

**Recommendation:** When washing by hand, use room-temperature water.

### Cover

Before cleaning and disinfecting, remove the cover from the cushion. Remove any hook fastener pieces from the bottom of the cover. Close the cover flap.

**To clean the cover:** Machine wash cold with mild detergent, gentle cycle; or wipe clean with neutral detergent and water. Tumble dry low or air dry.

**To disinfect the cover:** Hand wash, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes, and then rinse thoroughly. Or machine wash in hot water (60°C). Tumble dry low or air dry.

### Cushion and Foam Components

⚠ **Caution:** DO NOT machine wash or machine dry the cushions or foam.

**Before cleaning and disinfecting:** Remove the cover. Remove the cushion components from the foam base. (See the "Cover and Cushion Components" section in this manual.)

### Large Air Pad and IT Pad

⚠ **Cautions:** Use of the following may cause damage to the cushion components: abrasives (steel wool, scouring pads); caustic, automatic dishwasher detergents; cleaning products that contain petroleum or organic solvents, including acetone, toluene, Methyl Ethyl Ketone (MEK), naphtha, dry cleaning fluid, adhesive remover; oil-based lotions, lanolin; ozone gas; ultraviolet light cleaning methods. DO NOT expose the cushion or foam components to direct sunlight. Rinse thoroughly; detergent residue may cause the air cells to stick together.

⚠ **Warning:** DO NOT allow water or cleaning solution to enter the cushion components. DO NOT submerge the tubing in any liquid, including water.

**To clean the cushion components:** Hand wash, using mild liquid hand soap, hand dishwashing liquid, laundry detergent, or a multipurpose detergent (follow product label instructions.) Gently scrub all cushion surfaces, using a soft plastic bristle brush, a sponge or a cloth. Rinse with clean water. Air dry.

**To disinfect the cushion components:** Repeat cleaning instructions, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. Rinse thoroughly with clean water. Air dry.

⚠ **Caution:** Rinse thoroughly. Detergent residue may cause air cells to stick together.

### Foam Base

⚠ **Warning:** DO NOT submerge the foam in any liquid, including water.

⚠ **Warning:** Disinfectants are not effective on porous surfaces such as foam. If the foam becomes soiled or contaminated, DO NOT use with multiple individuals. Replace soiled or contaminated foam.

**To clean the foam:** To clean, wipe the foam gently with a cloth dampened with water and a household detergent, then wipe again with a clean, slightly damp cloth. Allow the foam to air dry thoroughly before putting the cover on.

### Care Symbols

Hand wash.

Wipe with a damp cloth.

Machine wash hot, normal, at the temperature shown.

Do not machine dry.

Tumble dry low

Line dry in the shade.

Machine wash gentle, cold

Do not machine wash.

Bleach (1 part bleach: 9 parts water)

Dry flat in the shade.

## Troubleshooting

For additional assistance, contact your equipment provider, distributor, or Customer Support.

Not holding air	<ul style="list-style-type: none"> <li>» Confirm that the inflation valve is completely closed.</li> <li>» Inspect all components for damage. Look for holes in the cushion. If very small holes or no holes are visible, follow the instructions in the Repair Kit provided with the product. For damage to the ISOFLO or the tubing or for large holes or leaks in the cushion components, see the Limited Warranty supplement, or contact Customer Support.</li> </ul> <p><b>⚠ Repair Kit Warning:</b> DO NOT allow any liquid to enter the cushion components.</p>
Not able to inflate or deflate the cushion	<ul style="list-style-type: none"> <li>» Confirm that the ISOFLO is open.</li> <li>» Confirm that the inflation valve is open.</li> <li>» Make sure that all quick disconnects are securely connected.</li> </ul>
Uncomfortable / unstable	<ul style="list-style-type: none"> <li>» Confirm that the air pad has the correct amount of air. See the Transition Check and Immersion Check instructions.</li> <li>» Make sure the cushion components are correctly placed on the foam base; the hook / loop fastener strips on the air cells and the foam base should match up.</li> <li>» Make sure the air cells face up and the top of the cover faces up.</li> <li>» Confirm that the cushion is centered beneath the individual and that the cushion is properly sized to the individual and the wheelchair.</li> </ul> <p>After checking all of the above, allow the individual at least one hour to adjust to the cushion.</p>
The cushion slides on the wheelchair	<ul style="list-style-type: none"> <li>» Make sure the non-skid bottom of the cushion cover faces down.</li> <li>» Use directional labels on the cushion and on the cover to orient the cushion.</li> <li>» When possible, use hook and loop fasteners to further secure the cushion to the seating surface. Hook fasteners are available upon request.</li> </ul>
A component is damaged	Contact Customer Support for a replacement.

## Product Specifications

**Materials:** Large Air Pad / IT Pad: neoprene rubber; made without latex; Inflation valve: Zytel; Tubing: urethane; ISOFLO Memory Control: polycarbonate, aluminum, polypropylene; Foam Base: polyurethane; Cover: two-way stretch, fluid-resistant top and sides, with a zipper on the back and sides, and a non-skid bottom. Refer to cover care label for material content and for compliance to flammability requirements.

WIDTH (W) - in. (cm) ⇄	14 (35.5)	15 (38)	16 (40.5)	17 (43)	18 (45.5)	20 (51)	<p><i>Notes:</i></p> <ul style="list-style-type: none"> <li>- All measurements are approximate.</li> <li>- Width and depth measurements are taken from the top of the foam base and are rounded to the nearest ¼ in. or 0.5 cm.</li> <li>- Item numbers and cover labeling indicate the wheelchair size that the product is designed to fit.</li> <li>- ** Depths shown include ¼ in. (1.5 cm) of foam base that fits between the wheelchair back posts.</li> </ul> <p><b>WEIGHT:</b> 4 lbs. / 1.8 kg (based on 1818)  <b>SEATED HEIGHT:</b> 3½ in. / 9 cm</p>
DEPTH (D)** - in. (cm) ⇄	Contoured Foam Base width x depth						
14¾ (37.5)	1414						
15¾ (40)		1515	1615				
16¾ (42.5)	1416	1516	1616	1716	1816	2016	
17¾ (45)		1517	1617	1717	1817	2017	
18¾ (47.5)			1618	1718	1818	2018	
20¾ (52.5)			1620	1720	1820	2020	

Cushion expected lifetime: 5 years

Cover expected lifetime: 1 year

## Storage, Transport, Disposal and Recycling

**Storage or Transport:** Clean and disinfect the product. Open the inflation valve. Place in a container that protects the product from moisture, contaminants and damage. After storing or transporting, inspect the product to ensure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

**Disposal:** The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incineration must be performed by a qualified, licensed waste management facility.

**Recycling:** Contact a local recycling agency to determine recycling options for the product.

## Limited Warranty

Limited Warranty Term from the date the product was originally purchased: cushion, 36 months; cover, 6 months. The warranty does not apply to punctures, tears or burns. See also the Limited Warranty supplement provided with the product, or contact Customer Support.

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