

ATTENTION!

PLEASE HANDLE WITH CARE

LOCK/UNLOCK - Use the lever(s) located on the wheelchair base to lock/unlock wheels. **USE CAUTION** as this will allow chair to roll freely. **Do not force or push** the chair when in lock mode.

When **lifting the wheelchair** is required only lift from the base. Wheelchair is bottom heavy.

DO NOT place chair on it's side. Sensitive components can be damaged.

ENSURE that main fuse is switched off.

Bring wheelchair to **cabin door**.

CONTACT AND CHAIR INFO

Contact Name: _____

Contact phone number: _____

Contact e-mail: _____

Chair model: _____ Colour(s) of chair: _____

Chair measurements: Weight: _____ Height: _____ Width: _____

Battery type: _____

Please refer to your owner's manual for additional safety tips and transportation guidelines.