


Accessories

Section 10



RIGID SIDE GUARD


TiLite™ TR, TRC, YR, TRA, YRA, Evo, BB, X, SX, XC, YG, YGS,
Aero X, 2GX, TX, Aero R, TR (SERIES 2), YR (SERIES 2)

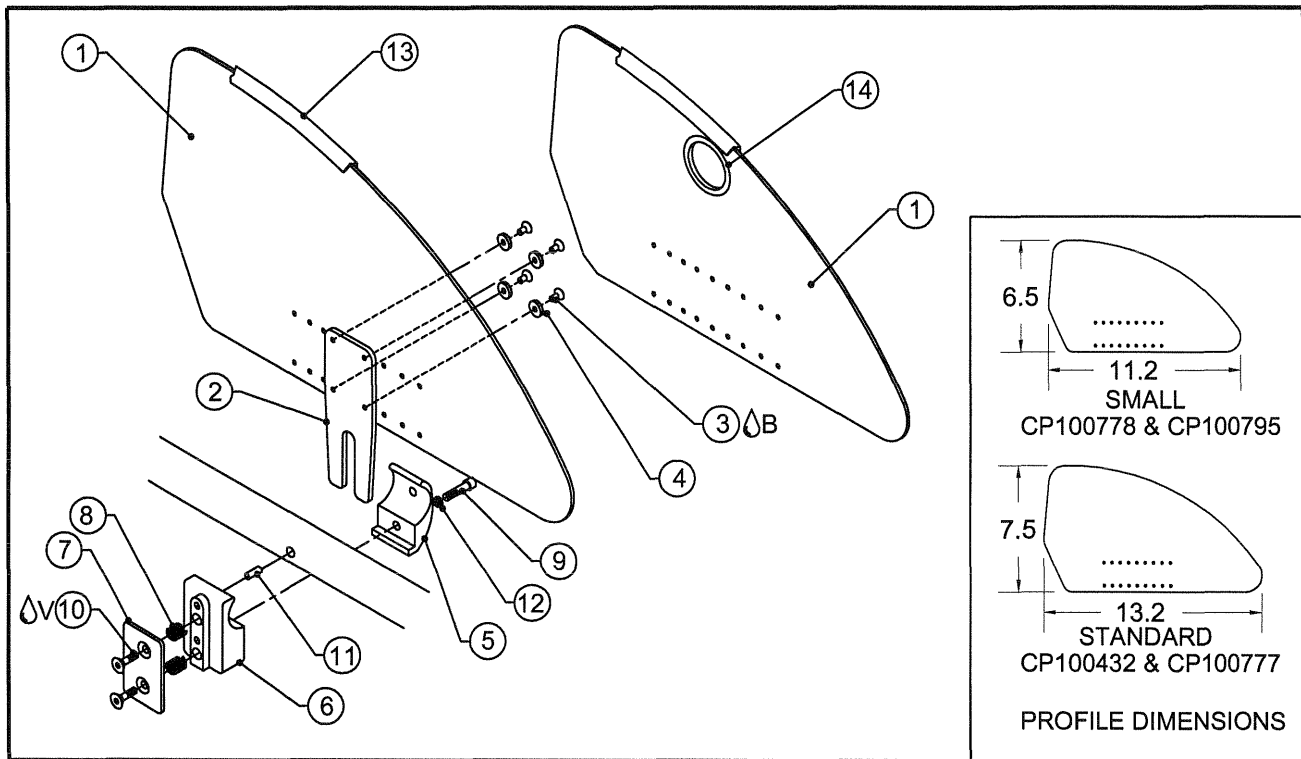

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100432	SIDE GUARD RIGID PANEL
1B	1	CP100778	SIDE GUARD PANEL SMALL
1C	1	CP100777	QUAD SIDE GUARD PANEL RIGID
1D	1	CP100795	QUAD SIDE GUARD PANEL SMALL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3ΔB	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	1	CP100434	1" TOE CLAMP
6	1	CP100436	SIDE GUARD MOUNT RIGID
7	1	CP100433	CAP FOR RIGID SIDE GUARD
8	2	HD100090	SPRING .266 OD X .196 ID X .310
9	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
10ΔV	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
11	1	HD100093	DOWEL PIN .250" OD X .500" LONG SS
12	1	HD100348	WASHER, .255ID x .395OD x .048
13	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
14	1	HD100151	RUBBER GROMMET 1-1/4" DIA
5-12	1	AS100106	SIDE GUARD CLAMP 1" ASSEMBLY
1A,2-4,13		AS100452	ALUM SIDE GUARD ASSEMBLY RIGHT
1A,2-4,13		AS100451	ALUM SIDE GUARD ASSEMBLY LEFT
1B,2-4,13		AS100389	SMALL ALUM SIDE GUARD ASSEMBLY RIGHT
1B,2-4,13		AS100390	SMALL ALUM SIDE GUARD ASSEMBLY LEFT
1C,2-4,13-14		AS100385	QUAD ALUM SIDE GUARD ASSEMBLY RIGHT
1C,2-4,13-14		AS100386	QUAD ALUM SIDE GUARD ASSEMBLY LEFT
1D,2-4,13-14		AS100387	QUAD SMALL ALUM SIDE GUARD ASSEMBLY RIGHT
1D,2-4,13-14		AS100388	QUAD SMALL ALUM SIDE GUARD ASSEMBLY LEFT
1A,2-13		AS100108	ALUM SIDE GUARD ASSEMBLY RIGHT W/ 1" CLAMP
1A,2-13		AS100107	ALUM SIDE GUARD ASSEMBLY LEFT W/ 1" CLAMP
1B,2-13		AS100383	SMALL ALUM SIDE GUARD ASSEMBLY RIGHT W/ 1" CLAMP
1B,2-13		AS100384	SMALL ALUM SIDE GUARD ASSEMBLY LEFT W/ 1" CLAMP
1C,2-14		AS100379	QUAD ALUM SIDE GUARD ASSEMBLY RIGHT W/ 1" CLAMP
1C,2-14		AS100380	QUAD ALUM SIDE GUARD ASSEMBLY LEFT W/ 1" CLAMP
1D,2-14		AS100381	QUAD SMALL ALUM SIDE GUARD ASSEMBLY RIGHT W/ 1" CLAMP
1D,2-14		AS100382	QUAD SMALL ALUM SIDE GUARD ASSEMBLY LEFT W/ 1" CLAMP

RIGID SIDE GUARD

TiLite™ ZR, ZRA, AERO Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

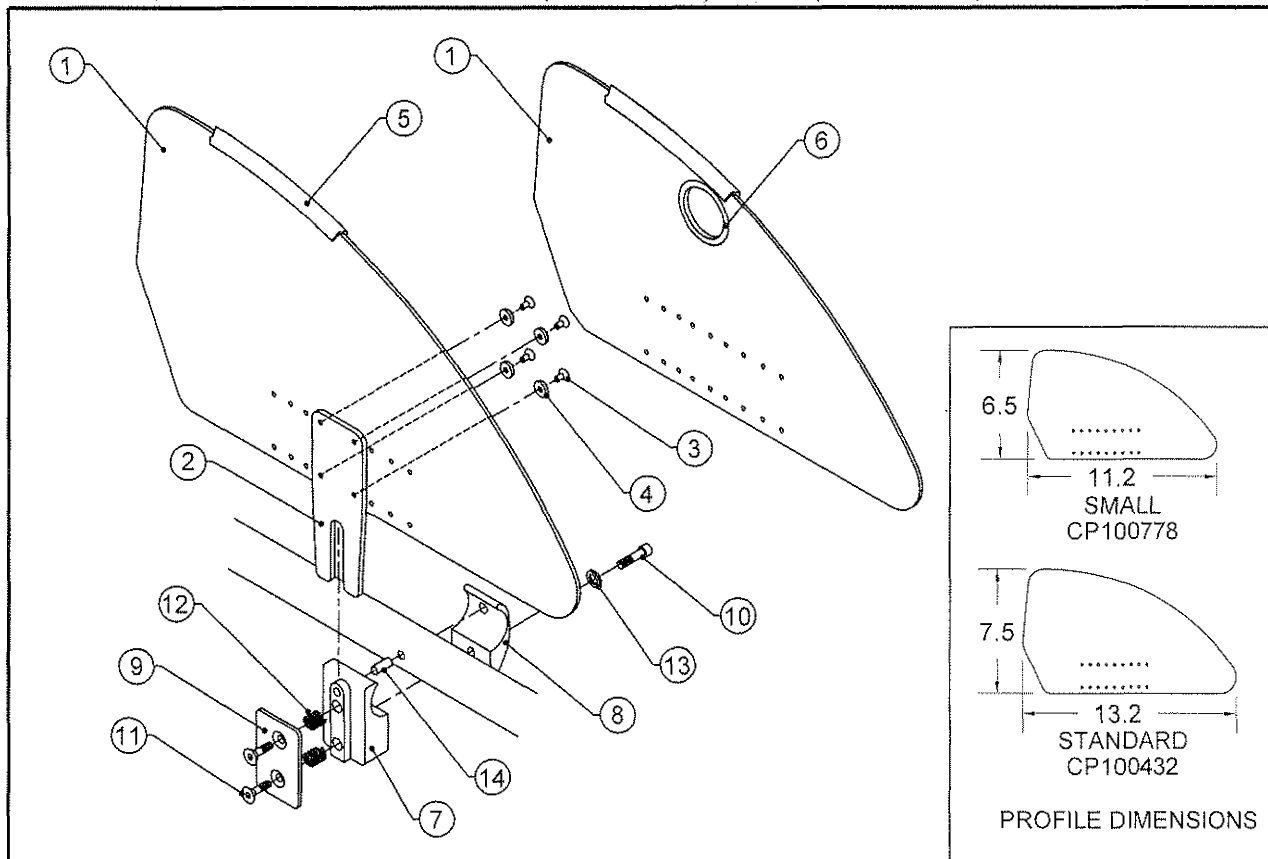


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100432	ALUMINUM SIDE GUARD PANEL
1B	1	CP100778	SMALL ALUMINUM SIDE GUARD PANEL
1C	1	CP100777	QUAD ALUMINUM SIDE GUARD PANEL
1D	1	CP100795	QUAD SMALL ALUMINUM SIDE GUARD PANEL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
6	1	HD100151	RUBBER GROMMET 1-1/4" DIA
7	1	CP102059	1-1/4" SIDE GUARD MOUNT RIGID
8	1	CP102296	TOE CLAMP 1 1/4 CAMBER MOUNT WITH HOLE
9	1	CP100433	CAP FOR RIGID SIDE GUARD
10	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
11	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
12	2	HD100090	SPRING .266 OD x .196 ID x .310"
13	1	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
14	1	HD100093	DOWEL PIN 1/4" X 1/2" STAINLESS STEEL
7-14		AS102333	SIDE GUARD CLAMP 1.25" ASSEMBLY
1A-5		AS100452	ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1A-5		AS100451	ALUMINUM SIDE GUARD ASSEMBLY LEFT
1B-5		AS100389	SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1B-5		AS100390	SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT
1C-6		AS100385	QUAD ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1C-6		AS100386	QUAD ALUMINUM SIDE GUARD ASSEMBLY LEFT
1D-6		AS100387	QUAD SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1D-6		AS100388	QUAD SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT
1A-5,7-14		AS102334	ALUMINUM SIDE GUARD ASSEMBLY RIGHT W/1.25" CLAMP
1A-5,7-14		AS102335	ALUMINUM SIDE GUARD ASSEMBLY LEFT W/1.25" CLAMP
1B-5,7-14		AS102336	SMALL ALUM SIDE GUARD ASSY RIGHT W/1.25" CLAMP
1B-5,7-14		AS102337	SMALL ALUM SIDE GUARD ASSY LEFT W/1.25" CLAMP
1C-14		AS102338	QUAD ALUM SIDE GUARD ASSY RIGHT W/1.25" CLAMP
1C-14		AS102339	QUAD ALUM SIDE GUARD ASSY LEFT W/1.25" CLAMP
1D-14		AS102340	QUAD SM ALUM SIDE GUARD ASSY RIGHT W/1.25" CLAMP
1D-14		AS102341	QUAD SM ALUM SIDE GUARD ASSY LEFT W/1.25" CLAMP



RIGID SIDE GUARD

TiLite™ TWIST

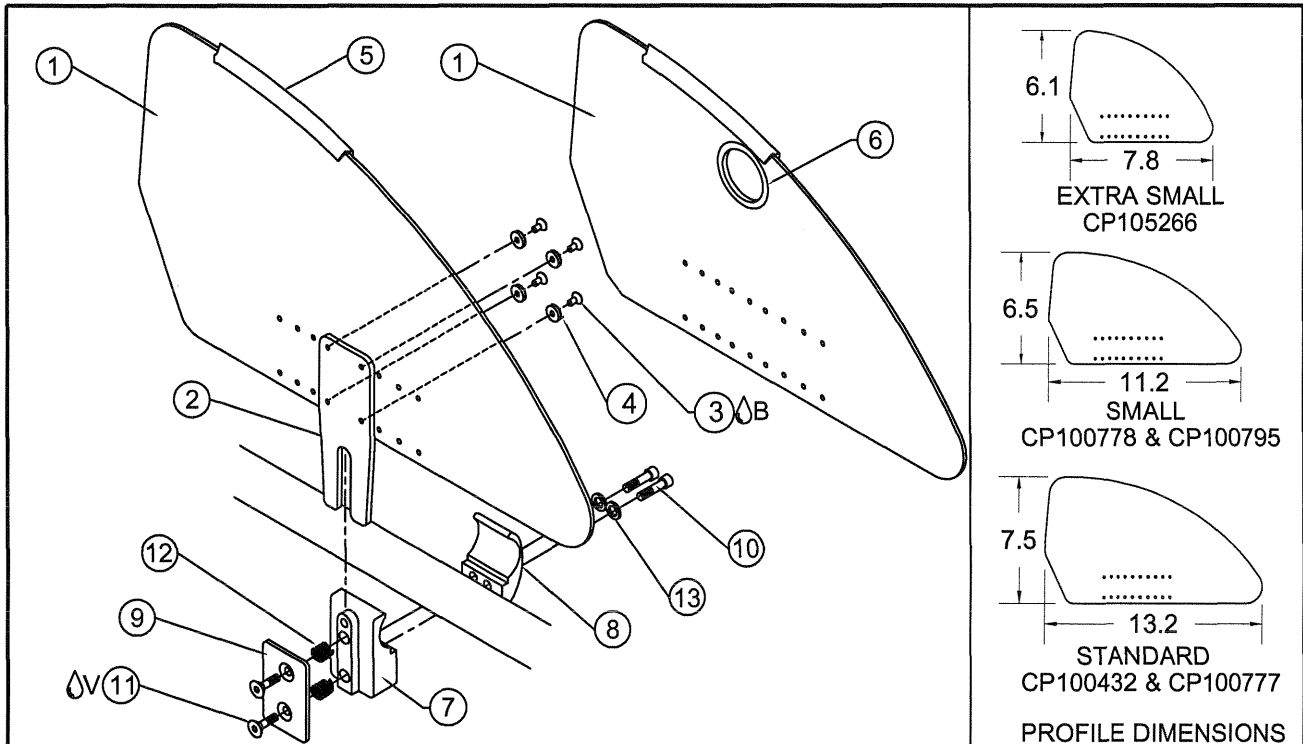


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100432	ALUMINUM SIDE GUARD PANEL
1B	1	CP100778	SMALL ALUMINUM SIDE GUARD PANEL
1C	1	CP105266	EXTRA SMALL ALUMINUM SIDE GUARD PANEL
1D	1	CP100777	QUAD ALUMINUM SIDE GUARD PANEL
1E	1	CP100795	QUAD SMALL ALUMINUM SIDE GUARD PANEL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3ΔB	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
6	1	HD100151	RUBBER GROMMET 1-1/4" DIA
7	1	CP105158	PED SIDE GUARD MOUNT
8	1	CP105159	PED SIDE GUARD CLAMP
9	1	CP100433	CAP FOR RIGID SIDE GUARD
10	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
11ΔV	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
12	2	HD100090	SPRING .266 OD x .196 ID x .310"
13	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
7-13		AS111875	SIDE GUARD CLAMP PED ASSEMBLY
1A-5		AS100452	ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1A-5		AS100451	ALUMINUM SIDE GUARD ASSEMBLY LEFT
1B-5		AS100389	SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1B-5		AS100390	SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT
1C-5		AS111855	EXTRA SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1C-5		AS111856	EXTRA SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT
1D-6		AS100385	QUAD ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1D-6		AS100386	QUAD ALUMINUM SIDE GUARD ASSEMBLY LEFT
1E-6		AS100387	QUAD SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1E-6		AS100388	QUAD SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT
1A-5,7-13		AS111857	ALUMINUM SIDE GUARD ASSEMBLY RIGHT W/ PED CLAMP
1A-5,7-13		AS111858	ALUMINUM SIDE GUARD ASSEMBLY LEFT W/PED CLAMP
1B-5,7-13		AS111859	SMALL ALUM SIDE GUARD ASSY RIGHT W/PED CLAMP
1B-5,7-13		AS111860	SMALL ALUM SIDE GUARD ASSY LEFT W/PED CLAMP
1C-5,7-13		AS111861	EXTRA SMALL ALUM SIDE GUARD ASSY RIGHT W/PED CLAMP
1C-5,7-13		AS111862	EXTRA SMALL ALUM SIDE GUARD ASSY LEFT W/PED CLAMP
1D-13		AS111863	QUAD ALUM SIDE GUARD ASSY RIGHT W/PED CLAMP
1D-13		AS111864	QUAD ALUM SIDE GUARD ASSY LEFT W/PED CLAMP
1E-13		AS111865	QUAD SM ALUM SIDE GUARD ASSY RIGHT W/PED CLAMP
1E-13		AS111866	QUAD SM ALUM SIDE GUARD ASSY LEFT W/PED CLAMP



RIGID SIDE GUARDS

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), Aero X (SERIES 2), 2GX (SERIES 2)

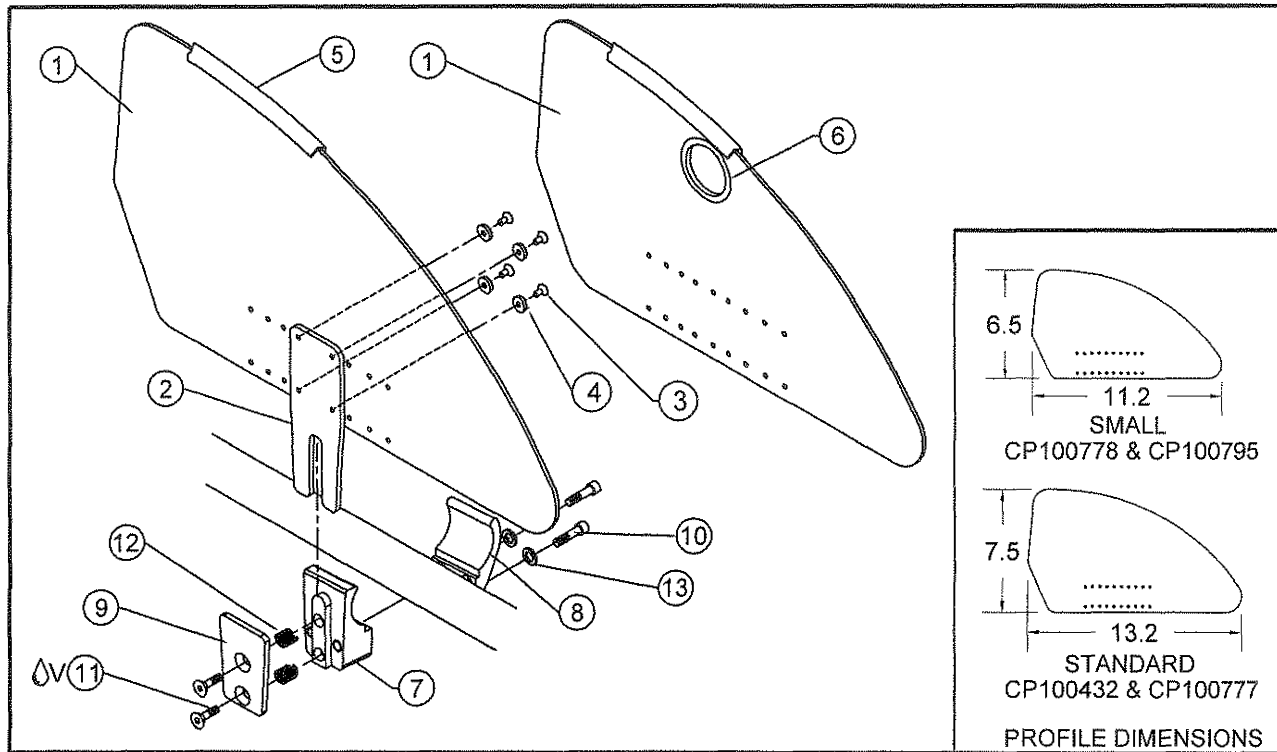

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100432	ALUMINUM SIDE GUARD PANEL
1B	1	CP100778	SMALL ALUMINUM SIDE GUARD PANEL
1C	1	CP100777	QUAD ALUMINUM SIDE GUARD PANEL
1D	1	CP100795	QUAD SMALL ALUMINUM SIDE GUARD PANEL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	36"	RW100291	EDGETRIM FOR 3/32 MATERIAL
6	1	HD100151	RUBBER GROMMET 1-1/4" DIA
7A	1	CP107407	ADJUSTABLE SIDE GUARD MOUNT (FOR Ø1.0" TUBE)
CON. 7B	1	CP107406	ADJUSTABLE SIDE GUARD MOUNT (FOR Ø1.25" TUBE)
8A	1	CP105159	ADJUSTABLE SIDE GUARD CLAMP (FOR Ø1.0" TUBE)
CON. 8B	1	CP107551	ADJUSTABLE SIDE GUARD CLAMP (FOR 1.25" TUBE)
9	1	CP107179	CAP FOR RIGID SIDEGUARD
10	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
11ΔV	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
12	2	HD100090	SPRING .266 OD x .196 ID x .310"
13	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
7A,8A,9-13		AS117123	SIDE GUARD CLAMP ASSEMBLY (FOR Ø1.0" TUBE)
7B,8B,9-13		AS117067	SIDE GUARD CLAMP ASSEMBLY (FOR Ø1.25" TUBE)
1A-5		AS100452	ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1A-5		AS100451	ALUMINUM SIDE GUARD ASSEMBLY LEFT
1B-5		AS100389	SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1B-5		AS100390	SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT
1C-6		AS100385	QUAD ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1C-6		AS100386	QUAD ALUMINUM SIDE GUARD ASSEMBLY LEFT
1D-6		AS100387	QUAD SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT
1D-6		AS100388	QUAD SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT

CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING
ASSEMBLIES ON NEXT PAGE



Parts Manual

RIGID SIDE GUARDS (CONT.)

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2), ZR (SERIES 2),
ZRA (SERIES 2), AERO Z (SERIES 2), Aero X (SERIES 2), 2GX (SERIES 2)

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A-5,7A,8A,9-13		AS117115	TAPERED ALUMINUM SIDE GUARD ASSEMBLY 1" RIGHT W/ CLAMP
1A-5,7A,8A,9-13		AS117116	TAPERED ALUMINUM SIDE GUARD ASSEMBLY 1" LEFT W/ CLAMP
1B-5,7A,8A,9-13		AS117117	TAPERED SMALL ALUM SIDE GUARD ASSY 1" RIGHT W/ CLAMP
1B-5,7A,8A,9-13		AS117118	TAPERED SMALL ALUM SIDE GUARD ASSY 1" LEFT W/ CLAMP
1C-6,7A,8A,9-13		AS117119	TAPERED QUAD ALUM SIDE GUARD ASSY 1" RIGHT W/ CLAMP
1C-6,7A,8A,9-13		AS117120	TAPERED QUAD ALUM SIDE GUARD ASSY 1" LEFT W/ CLAMP
1D-6,7A,8A,9-13		AS117121	TAPERED QUAD SM ALUM SIDE GUARD ASSY 1" RIGHT W/ CLAMP
1D-6,7A,8A,9-13		AS117122	TAPERED QUAD SM ALUM SIDE GUARD ASSY 1" LEFT W/ CLAMP
1A-5,7B,8B,9-13		AS117068	TAPERED ALUMINUM SIDE GUARD ASSEMBLY 1.25" RIGHT W/ CLAMP
1A-5,7B,8B,9-13		AS117069	TAPERED ALUMINUM SIDE GUARD ASSEMBLY 1.25" LEFT W/ CLAMP
1B-5,7B,8B,9-13		AS117070	TAPERED SMALL ALUM SIDE GUARD ASSY 1.25" RIGHT W/ CLAMP
1B-5,7B,8B,9-13		AS117071	TAPERED SMALL ALUM SIDE GUARD ASSY 1.25" LEFT W/ CLAMP
1C-6,7B,8B,9-13		AS117072	TAPERED QUAD ALUM SIDE GUARD ASSY 1.25" RIGHT W/ CLAMP
1C-6,7B,8B,9-13		AS117073	TAPERED QUAD ALUM SIDE GUARD ASSY 1.25" LEFT W/ CLAMP
1D-6,7B,8B,9-13		AS117074	TAPERED QUAD SM ALUM SIDE GUARD ASSY 1.25" RIGHT W/ CLAMP
1D-6,7B,8B,9-13		AS117075	TAPERED QUAD SM ALUM SIDE GUARD ASSY 1.25" LEFT W/ CLAMP


CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING



Parts Manual

CUSTOM RIGID SIDE GUARD


ALL TiLite™ CHAIRS WITHOUT HOLES IN FRAMES AND 1" TUBING

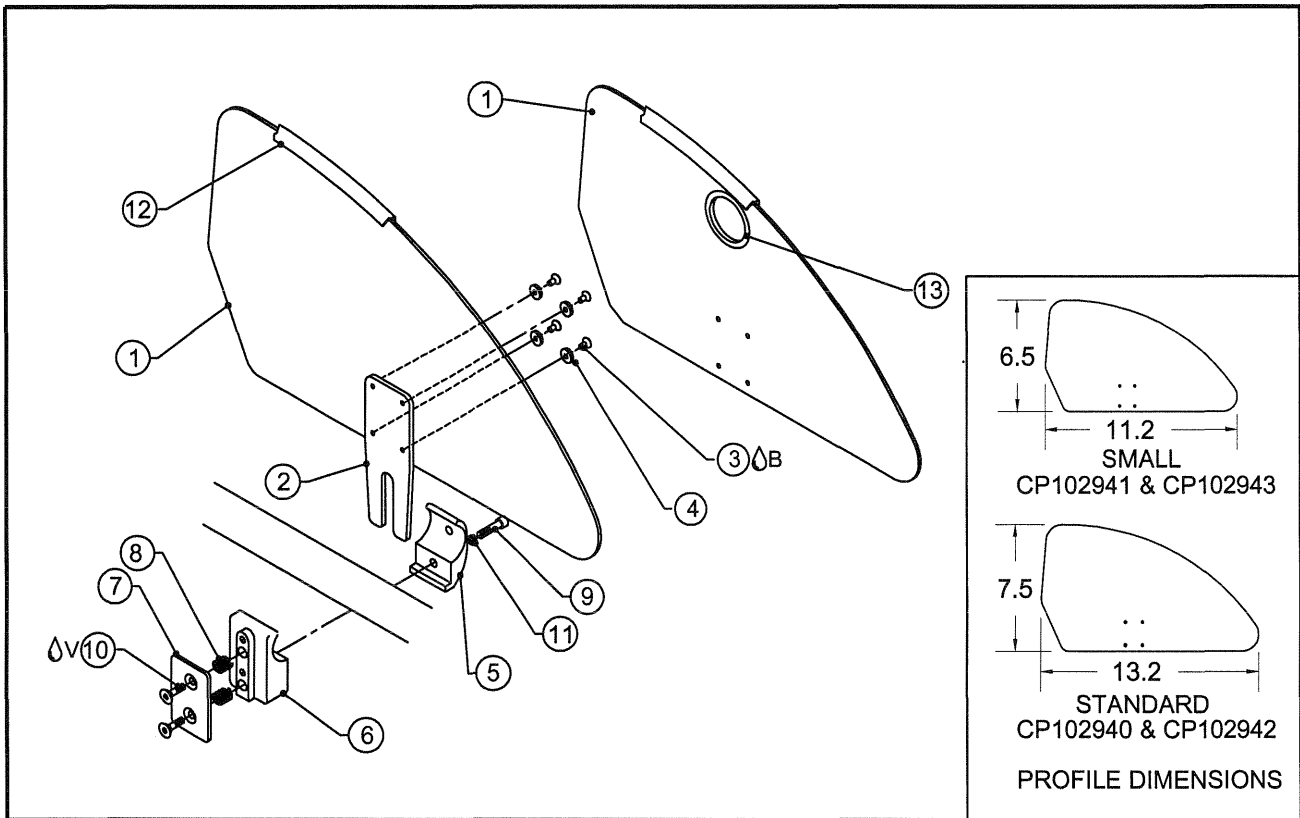

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100432	SIDE GUARD RIGID PANEL
1B	1	CP100778	SIDE GUARD PANEL SMALL
1C	1	CP100777	QUAD SIDE GUARD PANEL RIGID
1D	1	CP100795	QUAD SIDE GUARD PANEL SMALL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3ΔB	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	1	CP105774	1 INCH TOE CLAMP W/ GRIP PAINT
6	1	CP105773	SIDE GUARD MOUNT RIGID W/ GRIP PAINT
7	1	CP100433	CAP FOR RIGID SIDE GUARD
8	2	HD100090	SPRING .266 OD X .196 ID X .310
9	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
10ΔV	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
11	1	HD100348	WASHER, .255ID x .395OD x .048
12	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
13	1	HD100151	RUBBER GROMMET 1-1/4" DIA
5-11		AS102870	CUSTOM RIGID SIDE GUARD CLAMP 1" SUB-ASSY
1A,2-4,12		AS100452	ALUMINUM SIDE GUARD PANEL ASSY RIGHT
1A,2-4,12		AS100451	ALUMINUM SIDE GUARD PANEL ASSY LEFT
1B,2-4,12		AS100389	SMALL ALUMINUM SIDE GUARD PANEL RIGHT
1B,2-4,12		AS100390	SMALL ALUMINUM SIDE GUARD PANEL LEFT
1C,2-4,12-13		AS100385	QUAD ALUMINUM SIDE GUARD PANEL RIGHT
1C,2-4,12-13		AS100386	QUAD ALUMINUM SIDE GUARD PANEL LEFT
1D,2-4,12-13		AS100387	QUAD SMALL ALUMINUM SIDE GUARD PANEL RIGHT
1D,2-4,12-13		AS100388	QUAD SMALL ALUMINUM SIDE GUARD PANEL LEFT
1A,2-12		AS102871	ALUMINUM SIDE GUARD ASSEMBLY LEFT 1" FRAME
1A,2-12		AS102872	ALUMINUM SIDE GUARD ASSEMBLY RIGHT 1" FRAME
1B,2-12		AS102873	SMALL ALUMINUM SIDE GUARD ASSEMBLY RIGHT 1" FRAME
1B,2-12		AS102874	SMALL ALUMINUM SIDE GUARD ASSEMBLY LEFT 1" FRAME
1C,2-13		AS102875	QUAD ALUMINUM SIDE GUARD ASSEMBLY RIGHT 1" FRAME
1C,2-13		AS102876	QUAD ALUMINUM SIDE GUARD ASSEMBLY LEFT 1" FRAME
1D,2-13		AS102877	QUAD SMALL ALUMINUM SIDE GUARD ASSY RIGHT 1" FRAME
1D,2-13		AS102878	QUAD SMALL ALUMINUM SIDE GUARD ASSY LEFT 1" FRAME

CARBON FIBER SIDE GUARD

TiLite™ TR, TRC, YR, TRA, YRA, Evo, BB, X, SX, XC, YG, YGS,
Aero X, 2GX, TX, Aero R, TR (SERIES 2), YR (SERIES 2)

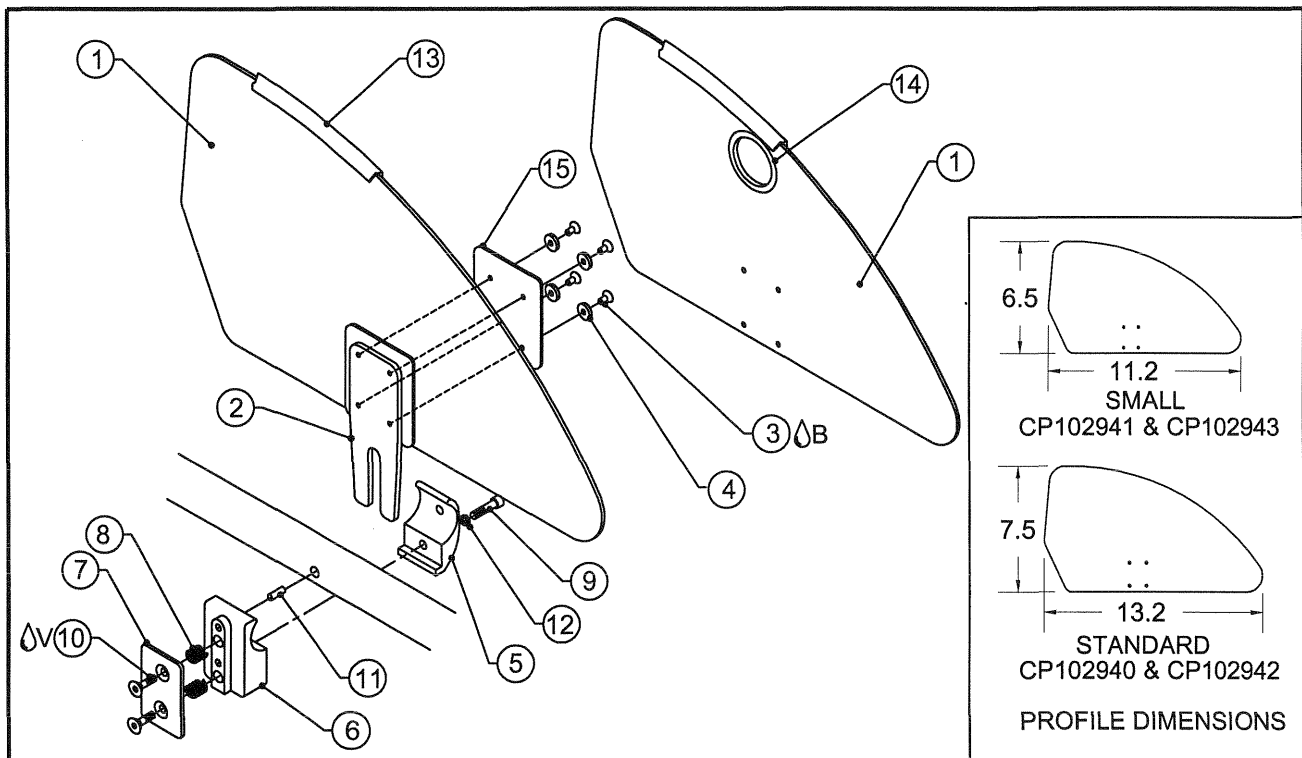


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP102940	CARBON FIBER SIDE GUARD PANEL
1B	1	CP102941	CARBON FIBER SIDE GUARD PANEL SMALL
1C	1	CP102942	QUAD CARBON FIBER SIDE GUARD PANEL
1D	1	CP102943	QUAD CARBON FIBER SIDE GUARD PANEL SMALL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3 Δ B	4	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK ZINC
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	1	CP100434	1" TOE CLAMP
6	1	CP100436	SIDE GUARD MOUNT RIGID
7	1	CP100433	CAP FOR RIGID SIDE GUARD
8	2	HD100090	SPRING .266 OD X .196 ID X .310
9	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
10 Δ V	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
11	1	HD100093	DOWEL PIN .250" OD X .500" LONG SS
12	1	HD100348	WASHER, .255ID x .395OD x .048
13	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
14	1	HD100151	RUBBER GROMMET 1-1/4" DIA
15	2	CP103182	CARBON SIDE GUARD REINFORCEMENT PLATE
5-12		AS100106	SIDE GUARD CLAMP SUB-ASSY
1A,2-4,13,15		AS103181	CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1A,2-4,13,15		AS103182	CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1B,2-4,13,15		AS103183	CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1B,2-4,13,15		AS103184	CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT
1C,2-4,12-15		AS103185	QUAD CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1C,2-4,12-15		AS103186	QUAD CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1D,2-4,12-15		AS103187	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1D,2-4,12-15		AS103188	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT
1A,2-13,15		AS103189	CARBON FIBER SIDE GUARD ASSEMBLY RIGHT W/ 1" CLAMP
1A,2-13,15		AS103190	CARBON FIBER SIDE GUARD ASSEMBLY LEFT W/ 1" CLAMP
1B,2-13,15		AS103191	SMALL CARBON FIBER SIDE GUARD ASSEMBLY RIGHT W/ 1" CLAMP
1B,2-13,15		AS103192	SMALL CARBON FIBER SIDE GUARD ASSEMBLY LEFT W/ 1" CLAMP
1C,2-14,15		AS103193	QUAD CARBON FIBER SIDE GUARD ASSY RIGHT W/ 1" CLAMP
1C,2-14,15		AS103194	QUAD CARBON FIBER SIDE GUARD ASSY LEFT W/ 1" CLAMP
1D,2-14,15		AS103195	QUAD SM CARBON FIBER SIDE GUARD ASSY RIGHT W/ 1" CLAMP
1D,2-14,15		AS103196	QUAD SM CARBON FIBER SIDE GUARD ASSY LEFT W/ 1" CLAMP



CARBON FIBER SIDE GUARD

TiLite™ ZR, ZRA, AERO Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

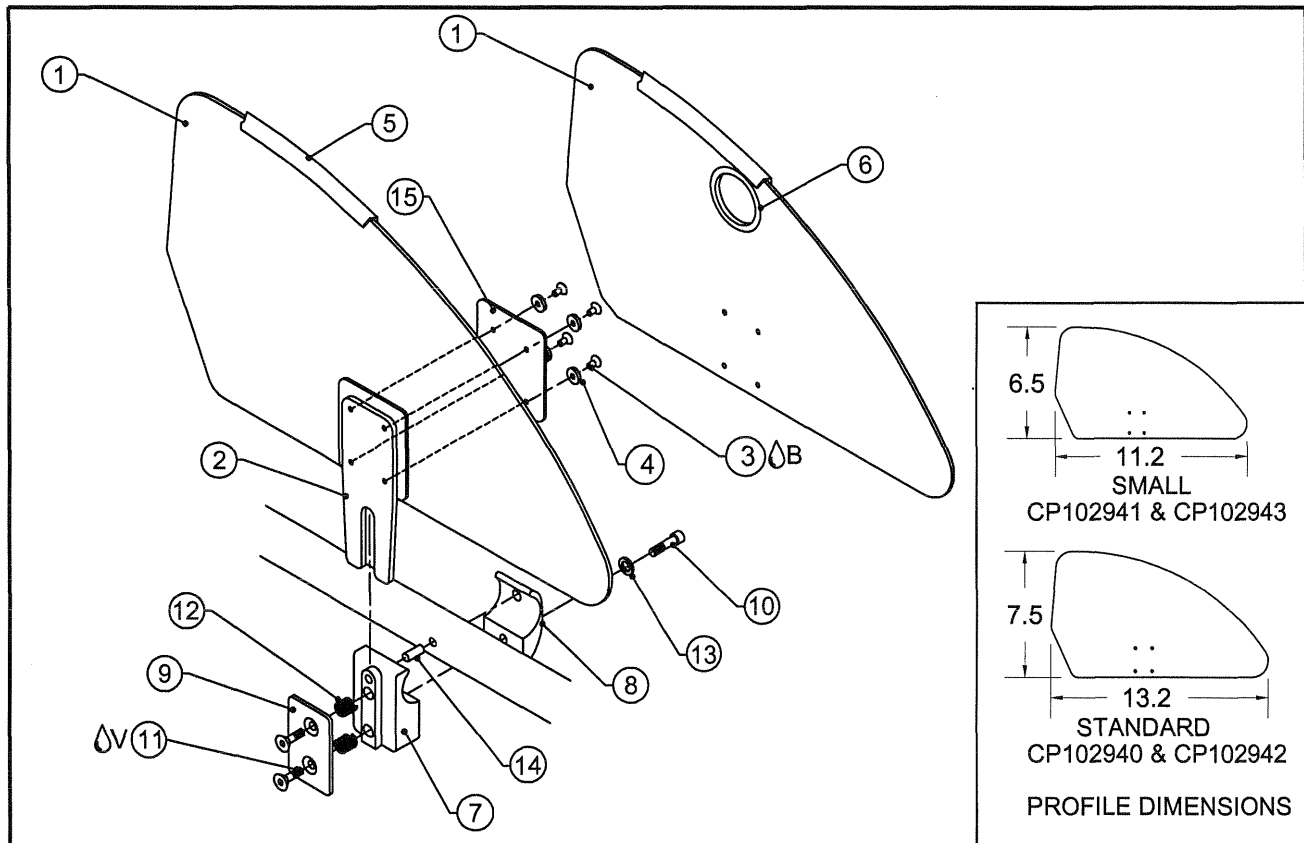


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP102940	CARBON FIBER SIDE GUARD PANEL
1B	1	CP102941	CARBON FIBER SIDE GUARD PANEL SMALL
1C	1	CP102942	QUAD CARBON FIBER SIDE GUARD PANEL
1D	1	CP102943	QUAD CARBON FIBER SIDE GUARD PANEL SMALL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3 Δ B	4	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK ZINC
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
6	1	HD100151	RUBBER GROMMET 1-1/4" DIA
7	1	CP102059	1-1/4" SIDE GUARD MOUNT RIGID
8	1	CP102296	TOE CLAMP 1 1/4 CAMBER MOUNT WITH HOLE
9	1	CP100433	CAP FOR RIGID SIDE GUARD
10	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
11 Δ V	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
12	2	HD100090	SPRING .266 OD x .196 ID x .310"
13	1	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
14	1	HD100460	DOWEL PIN 1/4" X 1-3/4" STAINLESS STEEL
15	2	CP103182	CARBON SIDE GUARD REINFORCEMENT PLATE
7-14		AS102333	SIDE GUARD CLAMP SUB-ASSEMBLY
1A-5,15		AS103181	CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1A-5,15		AS103182	CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1B-5,15		AS103183	CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1B-5,15		AS103184	CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT
1C-6,15		AS103185	QUAD CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1C-6,15		AS103186	QUAD CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1D-6,15		AS103187	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1D-6,15		AS103188	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT
1A-5,7-15		AS103216	CARBON FIBER SIDE GUARD ASSEMBLY RIGHT W/ 1.25" CLAMP
1A-5,7-15		AS103217	CARBON FIBER SIDE GUARD ASSEMBLY LEFT W/ 1.25" CLAMP
1B-5,7-15		AS103218	SMALL CARBON FIBER SIDE GUARD ASSY RIGHT W/ 1.25" CLAMP
1B-5,7-15		AS103219	SMALL CARBON FIBER SIDE GUARD ASSY LEFT W/ 1.25" CLAMP
1C-15		AS103220	QUAD CARBON FIBER SIDE GUARD ASSY RIGHT W/ 1.25" CLAMP
1C-15		AS103221	QUAD CARBON FIBER SIDE GUARD ASSY LEFT W/ 1.25" CLAMP
1D-15		AS103222	QUAD SM CARBON FIBER SIDE GUARD ASSY RT W/ 1.25" CLAMP
1D-15		AS103223	QUAD SM CARBON FIBER SIDE GUARD ASSY LT W/ 1.25" CLAMP



CARBON FIBER SIDE GUARDS

TiLite™ TWIST, TR (SERIES 3), AERO T, TRA (SERIES 2)
ZRA (SERIES 2), AERO Z (SERIES 2), Aero X (SERIES 2), 2GX (SERIES 2)

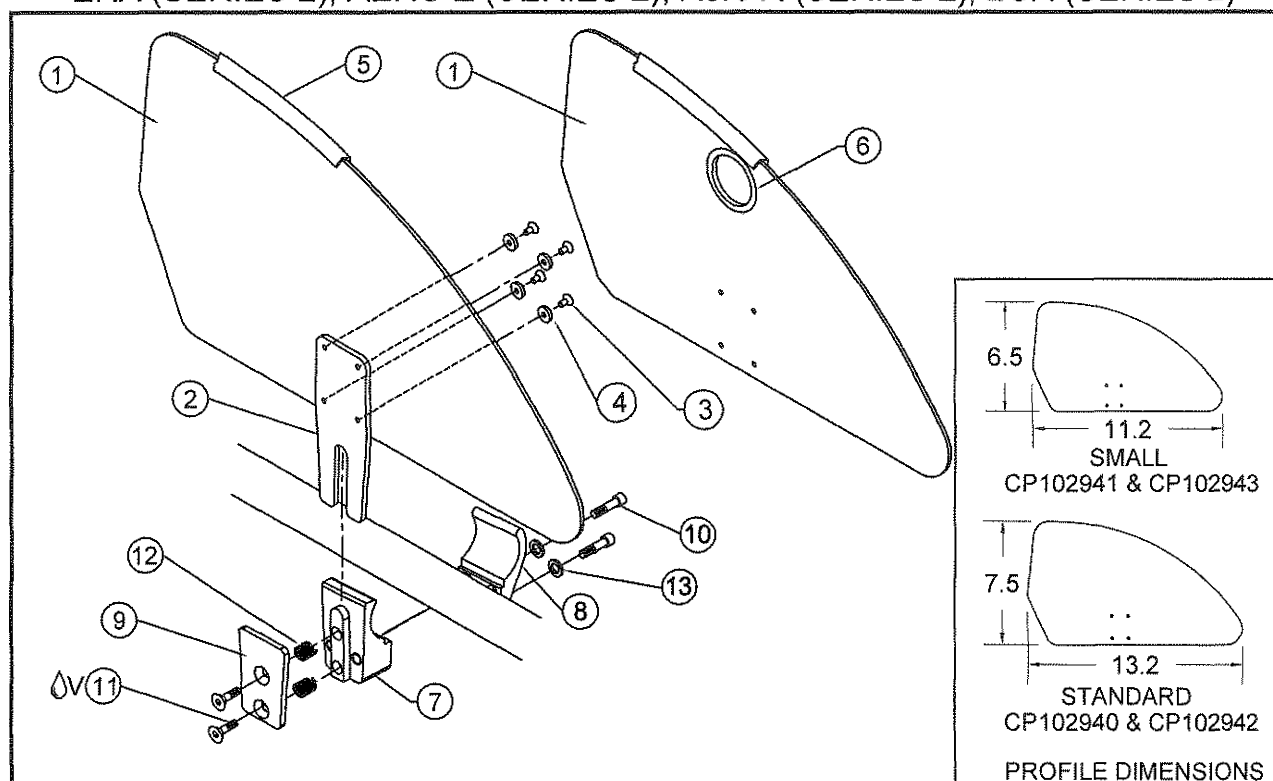

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP102940	CARBON FIBER SIDE GUARD PANEL
1B	1	CP102941	CARBON FIBER SIDE GUARD PANEL SMALL
1C	1	CP102942	QUAD CARBON FIBER SIDE GUARD PANEL
1D	1	CP102943	QUAD CARBON FIBER SIDE GUARD PANEL SMALL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3	4	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK ZINC
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	36"	RW100291	EDGETRIM FOR 3/32 MATERIAL
6	1	HD100151	RUBBER GROMMET 1-1/4" DIA
7A	1	CP107407	ADJUSTABLE SIDE GUARD MOUNT (FOR Ø1.0" TUBE)
CON. 7B	1	CP107406	ADJUSTABLE SIDE GUARD MOUNT (FOR Ø1.25" TUBE)
8A	1	CP105159	ADJUSTABLE SIDE GUARD CLAMP (FOR Ø1.0" TUBE)
CON. 8B	1	CP107551	ADJUSTABLE SIDE GUARD CLAMP (FOR Ø1.25" TUBE)
9	1	CP107179	CAP FOR RIGID SIDE GUARD
10	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
11 ΔV	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
12	2	HD100090	SPRING .266 OD X .196 ID X .310"
13	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
7A,8A,9-13		AS117123	SIDE GUARD CLAMP ASSEMBLY (FOR Ø1.0" TUBE)
7B,8B,9-13		AS117067	SIDE GUARD CLAMP ASSEMBLY (FOR Ø1.25" TUBE)
1A-5		AS103181	CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1A-5		AS103182	CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1B-5		AS103183	CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1B-5		AS103184	CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT
1C-6		AS103185	QUAD CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1C-6		AS103186	QUAD CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1D-6		AS103187	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1D-6		AS103188	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT

CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING
ASSEMBLIES ON NEXT PAGE



Parts Manual

CARBON FIBER SIDE GUARDS (CONT.)

TiLite™ TWIST, TR (SERIES 3), AERO T, TRA (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), Aero X (SERIES 2), 2GX (SERIES 2)




ITEM	QUANTITY	PART NO.	DESCRIPTION
1A-5,7A,8A,9-13		AS117568	TAPERED CARBON FIBER SIDE GUARD ASSEMBLY 1" RIGHT W/ CLAMP
1A-5,7A,8A,9-13		AS117569	TAPERED CARBON FIBER SIDE GUARD ASSEMBLY 1" LEFT W/ CLAMP
1B-5,7A,8A,9-13		AS117570	TAPERED SMALL CARBON FIBER SIDE GUARD ASSY 1" RIGHT W/ CLAMP
1B-5,7A,8A,9-13		AS117571	TAPERED SMALL CARBON FIBER SIDE GUARD ASSY 1" LEFT W/ CLAMP
1C-6,7A,8A,9-13		AS117572	TAPERED QUAD CARBON FIBER SIDE GUARD ASSY 1" RIGHT W/ CLAMP
1C-6,7A,8A,9-13		AS117573	TAPERED QUAD CARBON FIBER SIDE GUARD ASSY 1" LEFT W/ CLAMP
1D-6,7A,8A,9-13		AS117574	TAPERED QUAD SM CARBON FIBER SIDE GUARD ASSY 1" RIGHT W/ CLAMP
1D-6,7A,8A,9-13		AS117575	TAPERED QUAD SM CARBON FIBER SIDE GUARD ASSY 1" LEFT W/ CLAMP
1A-5,7B,8B,9-13		AS117576	TAPERED CARBON FIBER SIDE GUARD ASSEMBLY 1.25" RIGHT W/ CLAMP
1A-5,7B,8B,9-13		AS117577	TAPERED CARBON FIBER SIDE GUARD ASSEMBLY 1.25" LEFT W/ CLAMP
1B-5,7B,8B,9-13		AS117578	TAPERED SMALL CARBON FIBER SIDE GUARD ASSY 1.25" RIGHT W/ CLAMP
1B-5,7B,8B,9-13		AS117579	TAPERED SMALL CARBON FIBER SIDE GUARD ASSY 1.25" LEFT W/ CLAMP
1C-6,7B,8B,9-13		AS117580	TAPERED QUAD CARBON FIBER SIDE GUARD ASSY 1.25" RIGHT W/ CLAMP
1C-6,7B,8B,9-13		AS117581	TAPERED QUAD CARBON FIBER SIDE GUARD ASSY 1.25" LEFT W/ CLAMP
1D-6,7B,8B,9-13		AS117582	TAPERED QUAD SM CARBON FIBER SIDE GUARD ASSY 1.25" RIGHT W/ CLAMP
1D-6,7B,8B,9-13		AS117583	TAPERED QUAD SM CARBON FIBER SIDE GUARD ASSY 1.25" LEFT W/ CLAMP

CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING



CUSTOM CARBON FIBER SIDE GUARD

ALL TiLite™ CHAIRS WITHOUT HOLES IN FRAMES AND 1" TUBING

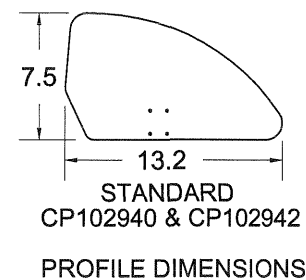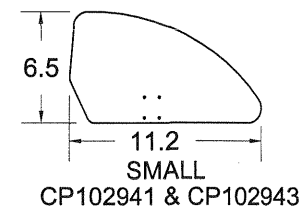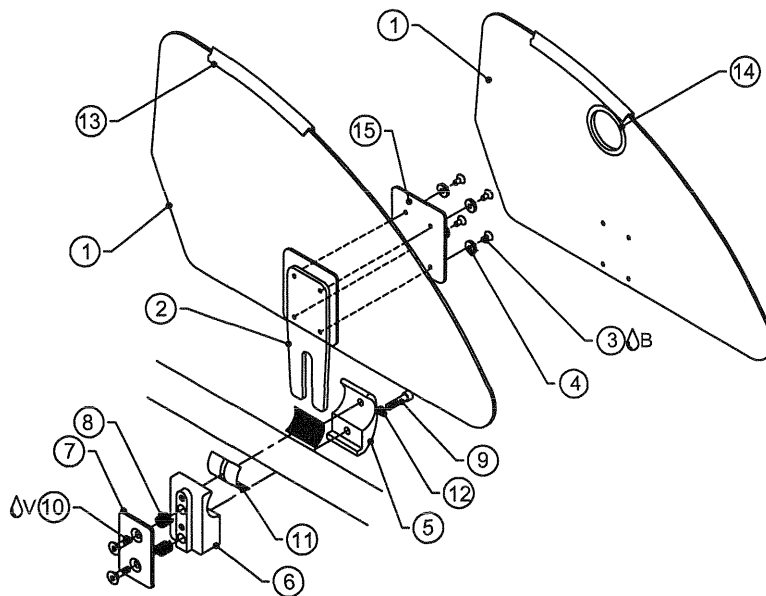


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP102940	CARBON FIBER SIDE GUARD PANEL
1B	1	CP102941	CARBON FIBER SIDE GUARD PANEL SMALL
1C	1	CP102942	QUAD CARBON FIBER SIDE GUARD PANEL
1D	1	CP102943	QUAD CARBON FIBER SIDE GUARD PANEL SMALL
2	1	CP100435	SIDE GUARD BRACKET RIGID
3ΔB	4	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK ZINC
4	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	1	CP100434	1 INCH TOE CLAMP
6	1	CP100436	SIDE GUARD MOUNT RIGID
7	1	CP100433	CAP FOR RIGID SIDE GUARD
8	2	HD100090	SPRING .266 OD X .196 ID X .310
9	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
10ΔV	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
11	2	CP102417	SIDE GUARD ANTI-SLIP TAPE
12	1	HD100348	WASHER, .255ID x .395OD x .048
13	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
14	1	HD100151	RUBBER GROMMET 1-1/4" DIA
15	2	CP103182	CARBON SIDE GUARD REINFORCEMENT PLATE
5-12		AS102870	CUSTOM SIDE GUARD CLAMP 1" SUB-ASSY
1A,2-4,13,15		AS103181	CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1A,2-4,13,15		AS103182	CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1B,2-4,13,15		AS103183	CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1B,2-4,13,15		AS103184	CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT
1C,2-4,13,15		AS103185	QUAD CARBON FIBER SIDE GUARD ASSEMBLY RIGHT
1C,2-4,13,15		AS103186	QUAD CARBON FIBER SIDE GUARD ASSEMBLY LEFT
1D,2-4,13,15		AS103187	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY RIGHT
1D,2-4,13,15		AS103188	QUAD CARBON FIBER SIDE GUARD SMALL ASSEMBLY LEFT
1A,2-13,15		AS103208	CUSTOM CARBON FIBER SIDE GUARD ASSEMBLY RIGHT W/ 1" CLAMP
1A,2-13,15		AS103209	CUSTOM CARBON FIBER SIDE GUARD ASSEMBLY LEFT W/ 1" CLAMP
1B,2-13,15		AS103210	CUSTOM SMALL CARBON FIBER SIDE GUARD ASSY RIGHT W/ 1" CLAMP
1B,2-13,15		AS103211	CUSTOM SMALL CARBON FIBER SIDE GUARD ASSY LEFT W/ 1" CLAMP
1C,2-14,15		AS103212	CUSTOM QUAD CARBON FIBER SIDE GUARD ASSY RIGHT W/ 1" CLAMP
1C,2-14,15		AS103213	CUSTOM QUAD CARBON FIBER SIDE GUARD ASSY LEFT W/ 1" CLAMP
1D,2-14,15		AS103214	CUSTOM QUAD SM CARBON FIBER SIDE GUARD ASSY RT W/ 1" CLAMP
1D,2-14,15		AS103215	CUSTOM QUAD SM CARBON FIBER SIDE GUARD ASSY LT W/ 1" CLAMP



FOLD DOWN SIDE GUARD

TiLite™ TR, TRC, YR, TRA, YRA, EVO, BB, YG, YGS
AERO R, ZR, ZRA, AERO Z, TR (SERIES 2), YR (SERIES 2),
ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

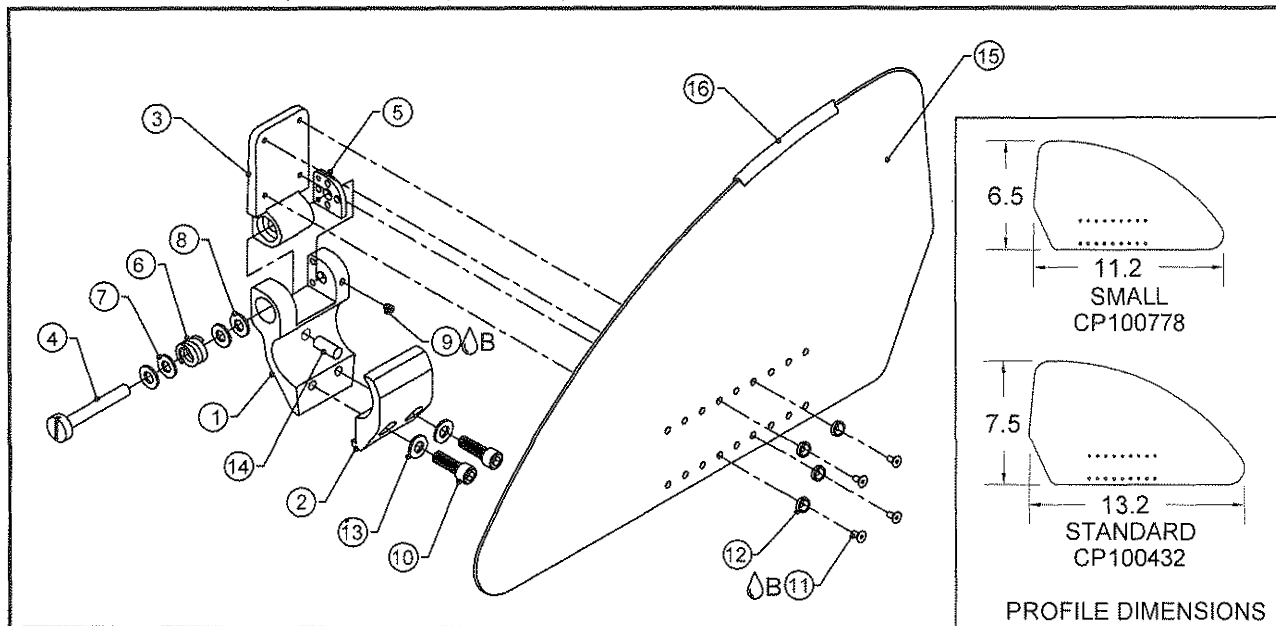


ITEM	QTY	PART NO.	DESCRIPTION
CON. 1A	1	CP102486	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.25" TUBE)
1B	1	CP102511	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.00" TUBE)
CON. 2A	1	CP102489	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.25" TUBE)
2B	1	CP102512	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.00" TUBE)
3	1	CP102490	FOLD DOWN SIDE GUARD BRACKET ASSEMBLY
4	1	CP102488	FOLD DOWN SIDE GUARD SCREW
5	1	CP102672	FOLD DOWN SIDE GUARD STRIKER PLATE - COMPOSITE
6	1	HD100379	SPRING - .546 OD .346 ID X .410 LG
7	2	HD100402	WASHER 1/4" LIGHT AN WASHER (.265 ID X .500 OD X .032 THK)
8	2	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
9ΔB	1	HD100132	8-32 X 3/16" SOCKET SET SCREW, NYLON TIP
10	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
11ΔB	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
12	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
13	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
14	1	HD100093	DOWEL PIN .250" OD X .500" LONG SS
15A	1	CP100778	SIDE GUARD PANEL SMALL
15B	1	CP100432	SIDE GUARD RIGID PANEL
16	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
1A,2A,3-14,15A,16		AS102914	ALUMINUM SMALL FOLD DOWN SIDE GUARD RIGHT ASSEMBLY, 1.25"
1A,2A,3-14,15A,16		AS102915	ALUMINUM SMALL FOLD DOWN SIDE GUARD LEFT ASSEMBLY, 1.25"
1A,2A,3-14,15B,16		AS102916	ALUMINUM STANDARD FOLD DOWN SIDE GUARD RIGHT ASSEMBLY, 1.25"
1A,2A,3-14,15B,16		AS102917	ALUMINUM STANDARD FOLD DOWN SIDE GUARD LEFT ASSEMBLY, 1.25"
1A,2A,3-14,15A,16		AS102918	ALUMINUM SMALL FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1.25"
1A,2A,3-14,15B,16		AS102919	ALUMINUM STANDARD FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1.25"
1B,2B,3-14,15A,16		AS102927	ALUMINUM SMALL FOLD DOWN SIDE GUARD RIGHT ASSEMBLY 1"
1B,2B,3-14,15A,16		AS102928	ALUMINUM SMALL FOLD DOWN SIDE GUARD LEFT ASSEMBLY 1"
1B,2B,3-14,15B,16		AS102929	ALUMINUM STANDARD FOLD DOWN SIDE GUARD RIGHT ASSEMBLY 1"
1B,2B,3-14,15B,16		AS102930	ALUMINUM STANDARD FOLD DOWN SIDE GUARD LEFT ASSEMBLY 1"
1B,2B,3-14,15A,16		AS102931	ALUMINUM SMALL FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1"
1B,2B,3-14,15B,16		AS102932	ALUMINUM STANDARD FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1"
1A,2A,3-10,13,14		AS102939	FOLD DOWN SIDE GUARD CLAMP ASSEMBLY, 1.25"
1B,2B,3-10,13,14		AS102940	FOLD DOWN SIDE GUARD CLAMP ASSEMBLY, 1"

CHAIR TYPE	SMALL RIGHT	SMALL LEFT	SMALL PAIR	STD RIGHT	STD LEFT	STD PAIR
TR,TRC,YR,TRA,YRA,EVO	AS102927	AS102928	AS102931	AS102929	AS102930	AS102932
ZR,ZRA,AEROZ	AS102914	AS102915	AS102918	AS102916	AS102917	AS102919

CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING

FOLD DOWN CARBON FIBER SIDE GUARD

TiLite™ TR, TRC, YR, TRA, YRA, EVO, BB, YG, YGS
AERO R, ZR, ZRA, AERO Z, TR (SERIES 2), YR (SERIES 2),
ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

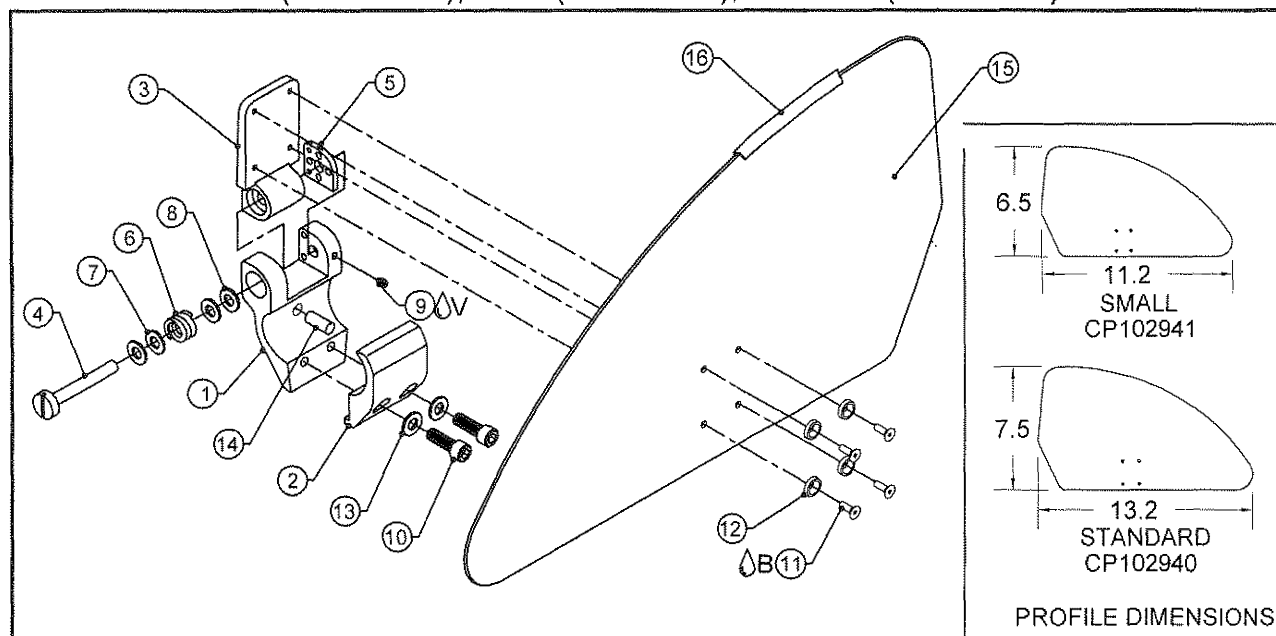

ITEM	QTY	PART NO.	DESCRIPTION
CON. 1A	1	CP102486	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.25" TUBE)
1B	1	CP102511	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.00" TUBE)
CON. 2A	1	CP102489	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.25" TUBE)
2B	1	CP102512	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.00" TUBE)
3	1	CP102490	FOLD DOWN SIDE GUARD BRACKET ASSEMBLY
4	1	CP102488	FOLD DOWN SIDE GUARD SCREW
5	1	CP102672	FOLD DOWN SIDE GUARD STRIKER PLATE - COMPOSITE
6	1	HD100379	SPRING - .546 OD .346 ID X .410 LG
7	2	HD100402	WASHER 1/4" LIGHT AN WASHER (.265 ID X .500 OD X .032 THK)
8	2	HD100067	.500 O.D. X .265 I.D. X .031 THICK WASHER
9ΔV	1	HD100132	8-32 X 3/16" SOCKET SET SCREW, NYLON TIP
10	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
11ΔB	4	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK ZINC
12	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
13	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
14	1	HD100093	DOWEL PIN .250" OD X .500" LONG SS
15A	1	CP102941	CARBON FIBER SIDE GUARD PANEL SMALL
15B	1	CP102940	CARBON FIBER SIDE GUARD PANEL
16	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
1A,2A,3-14,15A,16		AS103224	SMALL FOLD DOWN CARBON FIBER SIDE GUARD RIGHT ASSY W/ 1.25" CLAMP
1A,2A,3-14,15A,16		AS103225	SMALL FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSY W/ 1.25" CLAMP
1A,2A,3-14,15B,16		AS103226	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD RIGHT ASSY W/ 1.25" CLAMP
1A,2A,3-14,15B,16		AS103227	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSY W/ 1.25" CLAMP
1A,2A,3-14,15A,16		AS103228	SMALL FOLD DOWN CARBON FIBER SIDE GUARD ASSY W/ 1.25" CLAMP, PAIR
1A,2A,3-14,15B,16		AS103229	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD ASSY W/ 1.25" CLAMP, PAIR
1B,2B,3-14,15A,16		AS103230	SMALL FOLD DOWN CARBON FIBER SIDE GUARD RIGHT ASSY W/ 1" CLAMP
1B,2B,3-14,15A,16		AS103231	SMALL FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSY W/ 1" CLAMP
1B,2B,3-14,15B,16		AS103232	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD RIGHT ASSY W/ 1" CLAMP
1B,2B,3-14,15B,16		AS103233	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSY W/ 1" CLAMP
1B,2B,3-14,15A,16		AS103234	SMALL FOLD DOWN CARBON FIBER SIDE GUARD ASSY W/ 1" CLAMP, PAIR
1B,2B,3-14,15B,16		AS103235	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD ASSY W/ 1" CLAMP, PAIR
1A,2A,3-10,13,14		AS102939	FOLD DOWN SIDE GUARD 1.25" CLAMP ASSEMBLY
1B,2B,3-10,13,14		AS102940	FOLD DOWN SIDE GUARD 1" CLAMP ASSEMBLY


CHAIR TYPE	SMALL RIGHT	SMALL LEFT	SMALL PAIR	STD RIGHT	STD LEFT	STD PAIR
TR,TRC,YR,TRA,YRA,EVO	AS103230	AS103231	AS103234	AS103232	AS103233	AS103235
ZR,ZRA,AEROZ	AS103224	AS103225	AS103228	AS103226	AS103227	AS103229

CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING



FOLD DOWN ALUMINUM SIDE GUARD

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2)
ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

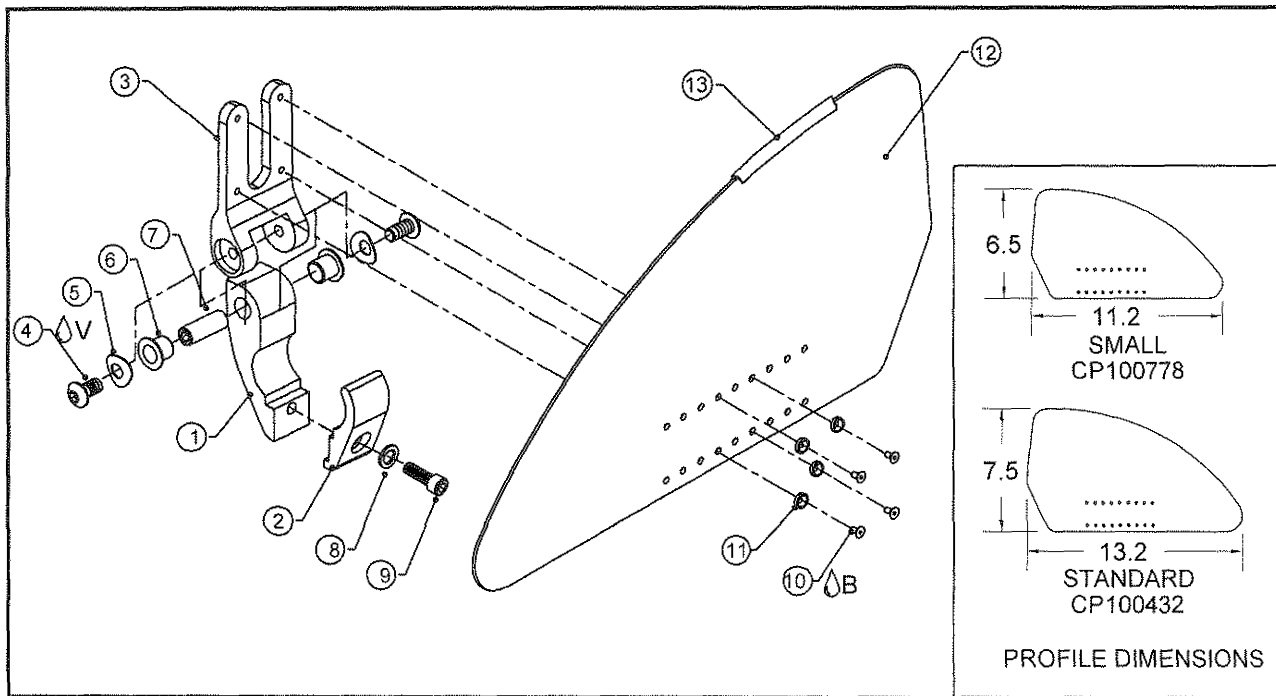

ITEM	QTY	PART NO.	DESCRIPTION
CON. 1A	1	CP107251	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.25" TUBE)
1B	1	CP107249	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.00" TUBE)
CON. 2A	1	CP107250	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.25" TUBE)
2B	1	CP107248	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.00" TUBE)
3	1	CP107272	FOLD DOWN SIDE GUARD BRACKET
4 V	2	HD100456	10-32 X .375" BUTTON HEAD CAP SCREW, BLACK ZINC PLATED
5	2	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
6	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
7	1	CP107273	FOLDING SIDEGUARD AXLE SLEEVE 10-32
8	1	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
9	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
10 B	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
11	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
12A	1	CP100778	SIDE GUARD PANEL SMALL
12B	1	CP100432	SIDE GUARD RIGID PANEL
13	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
1A, 2A, 3-11, 12A, 13		AS117016	ALUMINUM SMALL FOLD DOWN SIDE GUARD RIGHT ASSEMBLY 1.25"
1A, 2A, 3-11 12A, 13		AS117017	ALUMINUM SMALL FOLD DOWN SIDE GUARD LEFT ASSEMBLY 1.25"
1A, 2A, 3-11, 12B, 13		AS117018	ALUMINUM STANDARD FOLD DOWN SIDE GUARD RIGHT ASSEMBLY 1.25"
1A, 2A, 3-11, 12B, 13		AS117019	ALUMINUM STANDARD FOLD DOWN SIDE GUARD LEFT ASSEMBLY 1.25"
1A, 2A, 3-11, 12A, 13		AS117020	ALUMINUM SMALL FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1.25"
1A, 2A, 3-11, 12B, 13		AS117021	ALUMINUM STANDARD FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1.25"
1B, 2B, 3-11, 12A, 13		AS117029	ALUMINUM SMALL FOLD DOWN SIDE GUARD RIGHT ASSEMBLY 1"
1B, 2B, 3-11 12A, 13		AS117030	ALUMINUM SMALL FOLD DOWN SIDE GUARD LEFT ASSEMBLY 1"
1B, 2B, 3-11, 12B, 13		AS117031	ALUMINUM STANDARD FOLD DOWN SIDE GUARD RIGHT ASSEMBLY 1"
1B, 2B, 3-11, 12B, 13		AS117032	ALUMINUM STANDARD FOLD DOWN SIDE GUARD LEFT ASSEMBLY 1"
1B, 2B, 3-11, 12A, 13		AS117033	ALUMINUM SMALL FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1"
1B, 2B, 3-11, 12B, 13		AS117034	ALUMINUM STANDARD FOLD DOWN SIDE GUARD ASSEMBLY, PAIR 1"
1A, 2A, 3-9		AS117022	FOLD DOWN SIDE GUARD CLAMP ASSEMBLY, 1.25"
1B, 2B, 3-9		AS117035	FOLD DOWN SIDE GUARD CLAMP ASSEMBLY, 1"


CHAIR TYPE	SMALL RIGHT	SMALL LEFT	SMALL PAIR	STD RIGHT	STD LEFT	STD PAIR
TR, AERO T, TRA	AS117029	AS117030	AS117033	AS117031	AS117032	AS117034
ZR, ZRA, AEROZ	AS117016	AS117017	AS117020	AS117018	AS117019	AS117021

CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING



FOLD DOWN CARBON FIBER SIDE GUARD

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2)
ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

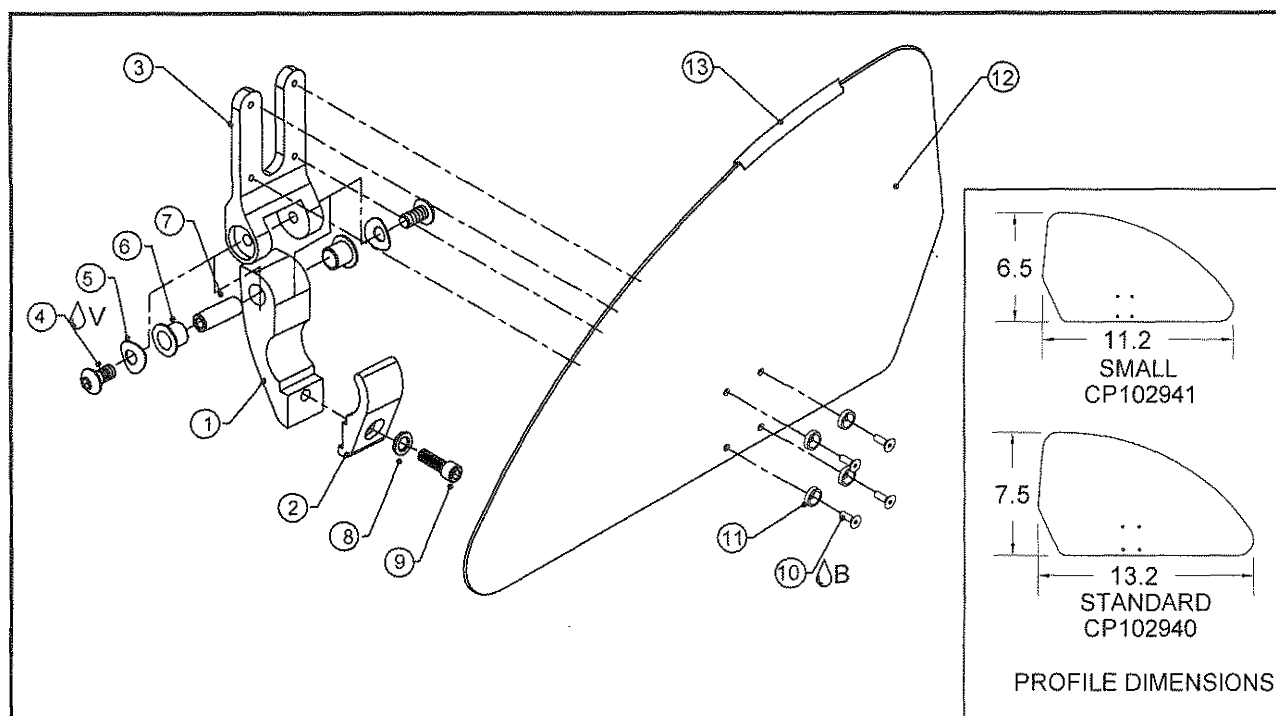

ITEM	QTY	PART NO.	DESCRIPTION
CON. 1A	1	CP107251	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.25" TUBE)
1B	1	CP107249	FOLD DOWN SIDE GUARD MOUNT (FOR Ø1.00" TUBE)
CON. 2A	1	CP107250	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.25" TUBE)
2B	1	CP107248	FOLD DOWN SIDE GUARD TOE CLAMP (FOR Ø1.00" TUBE)
3	1	CP107272	FOLD DOWN SIDE GUARD BRACKET
4△V	2	HD100456	10-32 X .375" BUTTON HEAD CAP SCREW, BLACK ZINC PLATED
5	2	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
6	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
7	1	CP107273	FOLDING SIDE GUARD AXLE SLEEVE 10-32
8	1	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
9	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
10△B	4	HD100444	6-32 X 3/8" FLAT HEAD CAP SCREW, BLACK
11	4	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
12A	1	CP102941	CARBON FIBER SIDE GUARD PANEL SMALL
12B	1	CP102940	CARBON FIBER SIDE GUARD PANEL
13	36"	RW100291	EDGE TRIM FOR 3/32 MATERIAL
1A,2A,3-11,12A,13		AS117023	SMALL FOLD DOWN CARBON FIBER SIDE GUARD RIGHT ASSEMBLY 1.25"
1A,2A,3-11,12A,13		AS117024	SMALL FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSEMBLY 1.25"
1A,2A,3-11,12B,13		AS117025	STANDARD FOLD CARBON FIBER DOWN SIDE GUARD RIGHT ASSEMBLY 1.25"
1A,2A,3-11,12B,13		AS117026	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSEMBLY 1.25"
1A,2A,3-11,12A,13		AS117027	SMALL FOLD DOWN CARBON FIBER SIDE GUARD ASSEMBLY, PAIR 1.25"
1A,2A,3-11,12B,13		AS117028	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD ASSEMBLY, PAIR 1.25"
1B,2B,3-11,12A,13		AS117036	SMALL FOLD DOWN CARBON FIBER SIDE GUARD RIGHT ASSY W/ 1" CLAMP
1B,2B,3-11,12A,13		AS117037	SMALL FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSY W/ 1" CLAMP
1B,2B,3-11,12B,13		AS117038	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD RIGHT ASSY W/ 1" CLAMP
1B,2B,3-11,12B,13		AS117039	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD LEFT ASSY W/ 1" CLAMP
1B,2B,3-11,12A,13		AS117040	SMALL FOLD DOWN CARBON FIBER SIDE GUARD ASSY W/ 1" CLAMP, PAIR
1B,2B,3-11,12B,13		AS117041	STANDARD FOLD DOWN CARBON FIBER SIDE GUARD ASSY W/ 1" CLAMP, PAIR
1A,2A,3-9		AS117022	FOLD DOWN SIDE GUARD 1.25" CLAMP ASSEMBLY
1B,2B,3-9		AS117035	FOLD DOWN SIDE GUARD 1" CLAMP ASSEMBLY


CHAIR TYPE	SMALL RIGHT	SMALL LEFT	SMALL PAIR	STD RIGHT	STD LEFT	STD PAIR
TR, AERO T, TRA	AS117036	AS117037	AS117040	AS117038	AS117039	AS117041
ZR,ZRA,AEROZ	AS117023	AS117024	AS117027	AS117025	AS117026	AS117028

CON. = CONSULT FACTORY FOR STYLE OF CLAMP WHEN ORDERING



REMOVABLE CARBON FIBER FENDERED SIDE GUARDS


TiLite™ TR (SERIES 2), YR (SERIES 2), Aero R

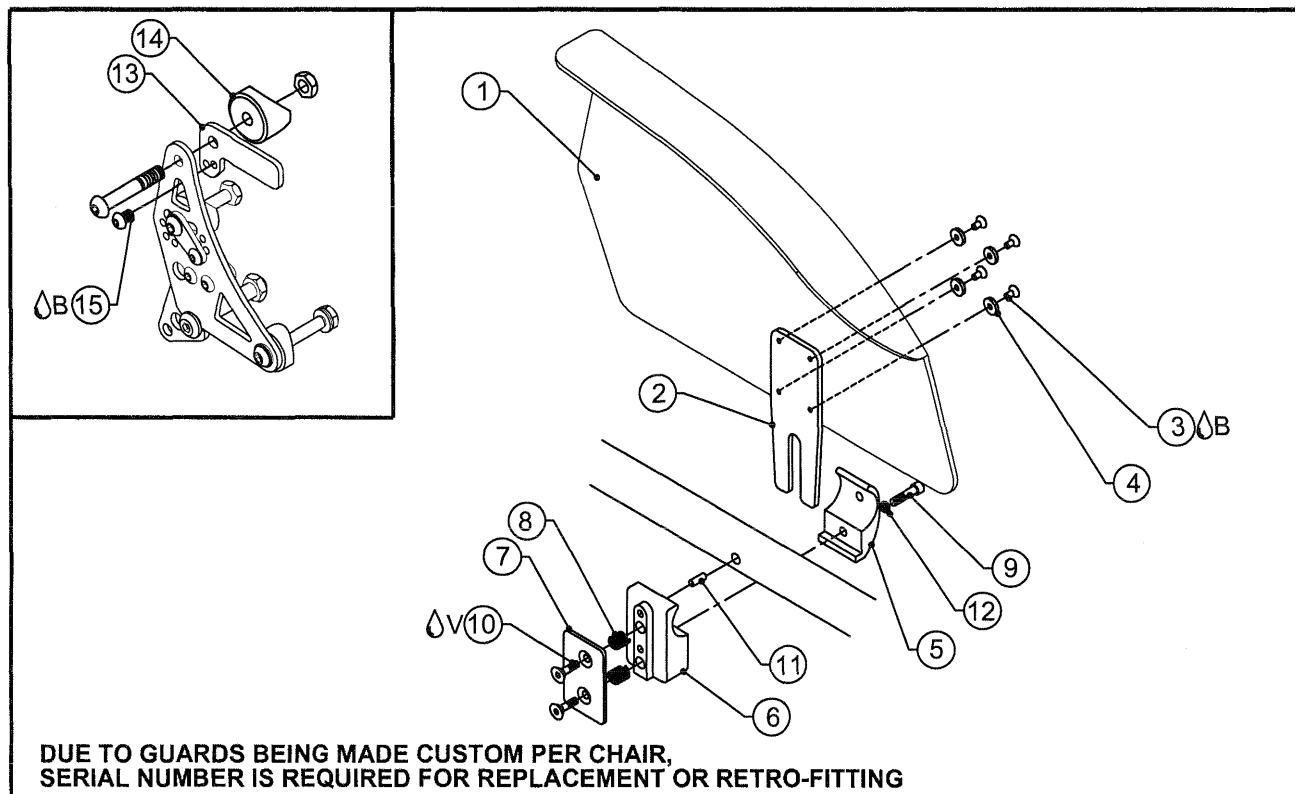

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	AS104899	CARBON FIBER FENDERED SIDE GUARD PAIR
2	2	CP100435	SIDEGUARD BRACKET RIGID
3ΔB	8	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK
4	8	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	2	CP100434	1" TOE CLAMP
6	2	CP100436	SIDEGUARD MOUNT RIGID
7	2	CP100433	CAP FOR RIGID SIDEGUARD
8	4	HD100090	SPRING .266 OD X .196 ID X .310
9	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
10ΔV	4	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
11	2	HD100093	DOWEL PIN .250" OD X .500" LONG SS
12	2	HD100348	WASHER, .255ID x .395OD x .048
13A	1	CP103516	FENDERED SIDEGUARD TAB LEFT
13B	1	CP103523	FENDERED SIDEGUARD TAB RIGHT
14	2	HD100014	COVED WASHER 1" OD X 1/4" ID X 1/8" THK
15ΔB	2	HD100069	10-32 X 1/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
5-12		AS100106	RIGID SIDEGUARD CLAMP SUB-ASSY
1-4		AS104900	CARBON FIBER FENDERED SIDEGUARD ASSEMBLY, PAIR
1-15		AS103892	CARBON FIBER FENDERED SIDEGUARD, W/ 1" CLAMP, PAIR

REMOVABLE CARBON FIBER FENDERED SIDE GUARDS


TiLite™ ZR, ZRA, AERO Z

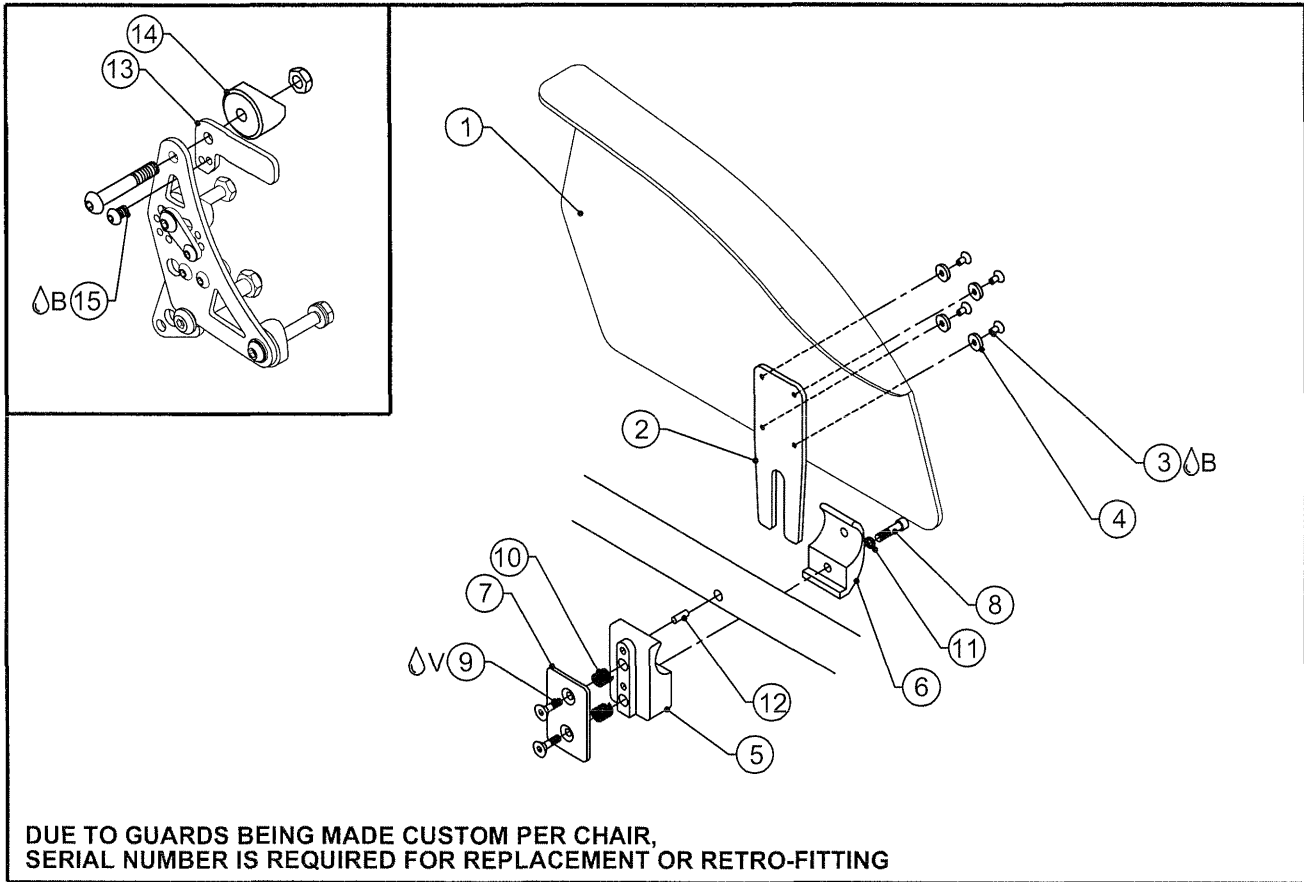

ITEM	QTY	PART NO.	DESCRIPTION
1	1	AS104899	CARBON FIBER FENDERED SIDE GUARD PAIR
2	2	CP100435	SIDEGUARD BRACKET RIGID
3B	8	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK
4	8	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	2	CP102059	1-1/4" SIDEGUARD MOUNT RIGID
6	2	CP102296	TOE CLAMP 1 1/4 CAMBER MOUNT WITH HOLE
7	2	CP100433	CAP FOR RIGID SIDEGUARD
8	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
9V	4	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
10	4	HD100090	SPRING .266 OD x .196 ID x .310"
11	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	2	HD100460	DOWEL PIN 1/4" X 1-3/4" STAINLESS STEEL
13A	1	CP103516	FENDERED SIDEGUARD TAB LEFT
13B	1	CP103523	FENDERED SIDEGUARD TAB RIGHT
14	2	HD100014	COVED WASHER 1" OD X 1/4" ID X 1/8" THK
15B	2	HD100069	10-32 X 1/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
5-12		AS102333	SIDEGUARD CLAMP ASSEMBLY 1.25"
1-4		AS104900	CARBON FIBER FENDERED SIDEGUARD ASSEMBLY, PAIR
1-15		AS103893	CARBON FIBER FENDERED SIDEGUARD, W/ 1/4" CLAMP, PAIR

REMOVABLE CARBON FIBER FENDERED SIDE GUARDS

TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

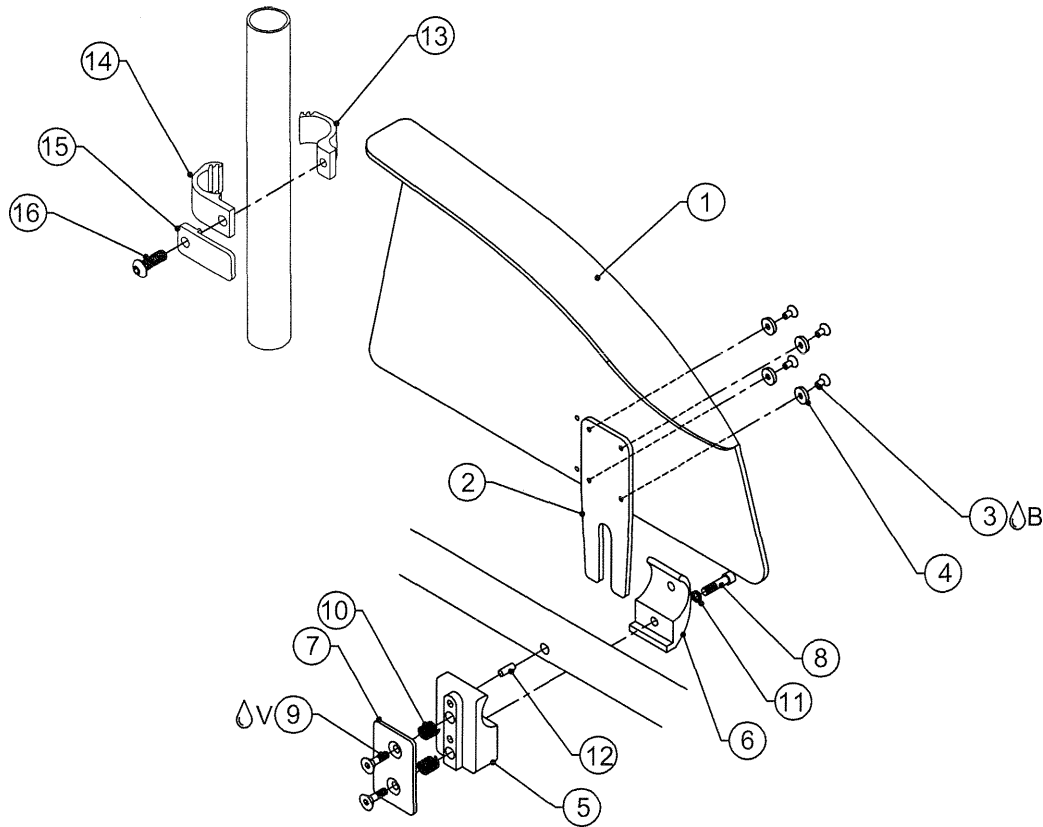


**DUE TO GUARDS BEING MADE CUSTOM PER CHAIR,
SERIAL NUMBER IS REQUIRED FOR REPLACEMENT OR RETRO-FITTING**

ITEM	QTY	PART NO.	DESCRIPTION
1	1	AS104899	CARBON FIBER FENDERED SIDE GUARD PAIR
2	2	CP100435	SIDEGUARD BRACKET RIGID
3 ^B	8	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK
4	8	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	2	CP102059	1-1/4" SIDEGUARD MOUNT RIGID
6	2	CP102296	TOE CLAMP 1 1/4 CAMBER MOUNT WITH HOLE
7	2	CP100433	CAP FOR RIGID SIDEGUARD
8	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
9 ^V	4	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
10	4	HD100090	SPRING .266 OD x .196 ID x .310"
11	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	2	HD100460	DOWEL PIN 1/4" X 1-3/4" STAINLESS STEEL
13	2	CP103920	BAND CLAMP, THREADED
14	2	CP103919	BAND CLAMP, THRU HOLE
15	2	CP104720	FENDERED SIDE GUARD TAB
16	2	HD100408	1/4-28 X 7/8" BUTTON HEAD CAP SCREW BLACK ZINC
5-12		AS102333	SIDEGUARD CLAMP ASSEMBLY 1.25"
1-4		AS104900	CARBON FIBER FENDERED SIDEGUARD ASSEMBLY, PAIR
1-16		AS110970	CARBON FIBER FENDERED SIDEGUARD, W/ 1 1/4" CLAMP, PAIR

REMOVABLE CARBON FIBER FENDERED SIDE GUARDS

TiLite™ TWIST, TR (SERIES 3), AERO T, TRA (SERIES 2)

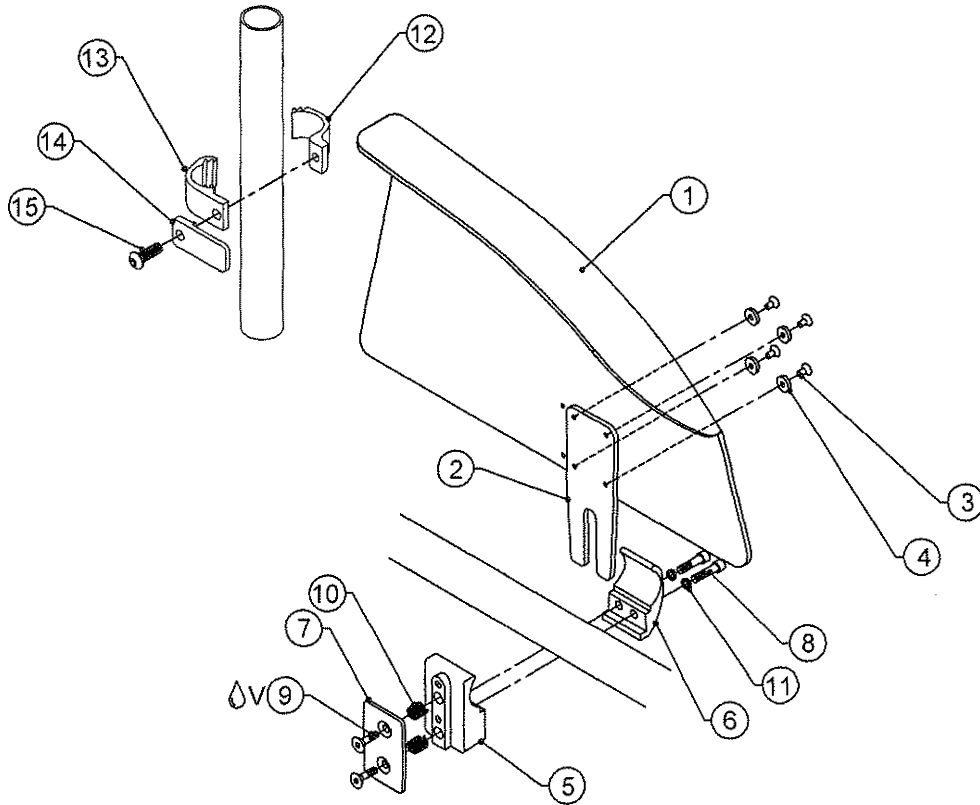


DUE TO GUARDS BEING MADE CUSTOM PER CHAIR,
SERIAL NUMBER IS REQUIRED FOR REPLACEMENT OR RETRO-FITTING

ITEM	QTY	PART NO.	DESCRIPTION
1	1	AS104899	CARBON FIBER FENDERED SIDE GUARD PAIR
2	2	CP100435	SIDE GUARD BRACKET RIGID
3	8	HD100466	6-32 X 7/16" FLAT HEAD CAP SCREW, BLACK
4	8	HD100449	NYLON COUNTERSUNK WASHER #6, .437 O.D. X .1" THICK
5	2	CP105158	ADJUSTABLE SIDE GUARD MOUNT
6	2	CP105159	ADJUSTABLE SIDE GUARD CLAMP
7	2	CP100433	CAP FOR RIGID SIDE GUARD
8	4	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK
9 Δ V	4	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
10	4	HD100090	SPRING .266 OD x .196 ID x .310"
11	4	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	2	CP103920	BAND CLAMP, THREADED
13	2	CP103919	BAND CLAMP, THRU HOLE
14	2	CP104720	FENDERED SIDE GUARD TAB
15	2	HD100408	1/4-28 X 7/8" BUTTON HEAD CAP SCREW BLACK ZINC
5-11		AS111875	ADJUSTABLE SIDE GUARD CLAMP ASSEMBLY
1-4		AS104900	CARBON FIBER FENDERED SIDE GUARD ASSEMBLY, PAIR
1-15		AS111876	CARBON FIBER FENDERED SIDE GUARD, W/ ADJ CLAMP, PAIR

ALUMINUM FENDER SIDE GUARDS


TiLite™ 2GX, AERO X

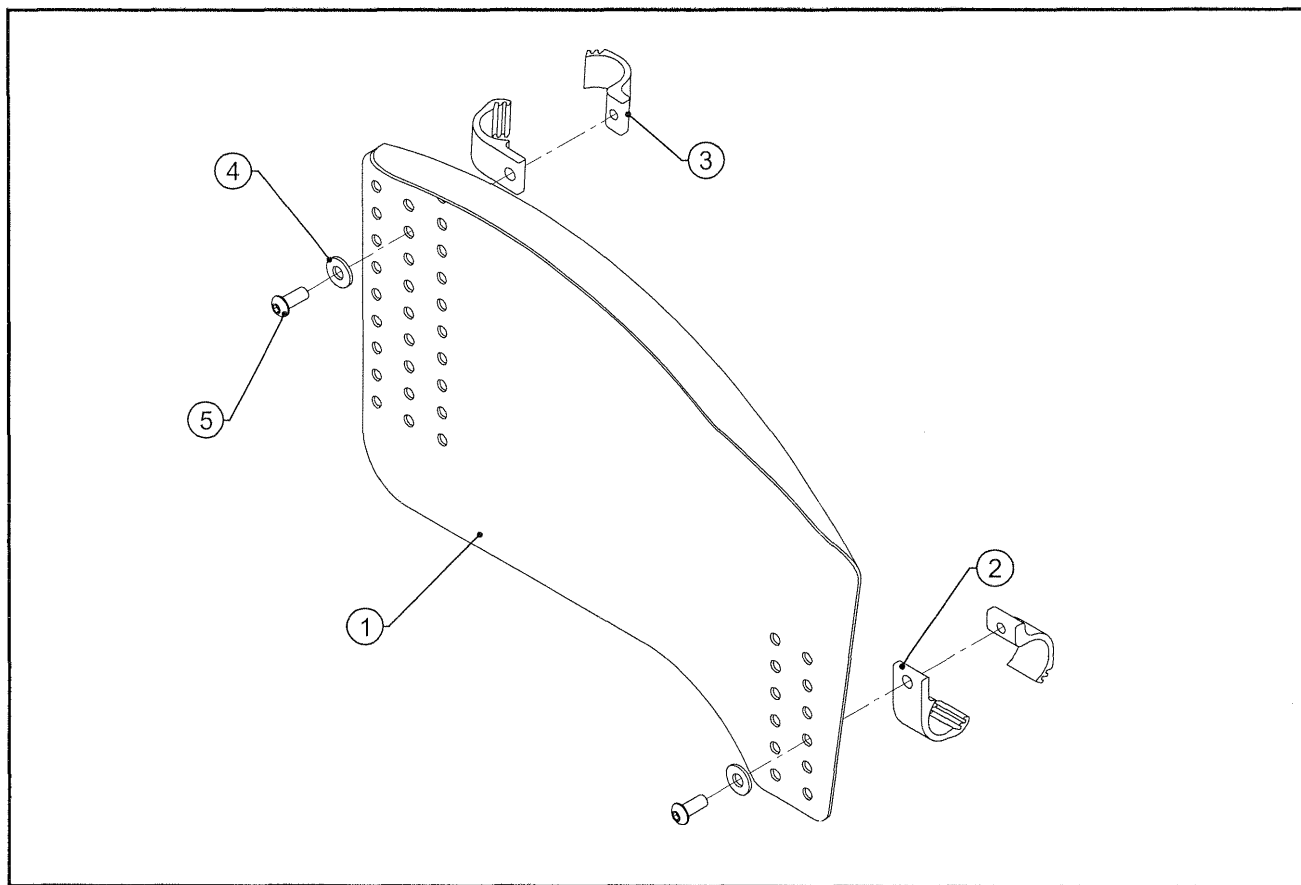

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103560	ALUMINUM FIXED FENDERED SIDE GUARD RIGHT
1B	1	CP103561	ALUMINUM FIXED FENDERED SIDE GUARD LEFT
2	2	CP103919	BAND CLAMP, THRU HOLE
3	2	CP103920	BAND CLAMP, THREADED
4	2	HD100454	1/4 SAE FLAT WASHER, BLACK ZINC PLATED
5	2	HD100531	1/4-28 X 5/8" BUTTON HEAD CAP SCREW, BLACK ZINC
1A,2-5		AS103799	ALUMINUM FENDER SIDE GUARD, RIGHT
1B,2-5		AS103800	ALUMINUM FENDER SIDE GUARD, LEFT
1A,1B,2-5		AS104544	ALUMINUM FENDER SIDE GUARD, PAIR

ALUMINUM ADJUSTABLE FENDERED SIDE GUARD ASSEMBLY

TiLite™ AERO X (SERIES 2), 2GX (SERIES 2)

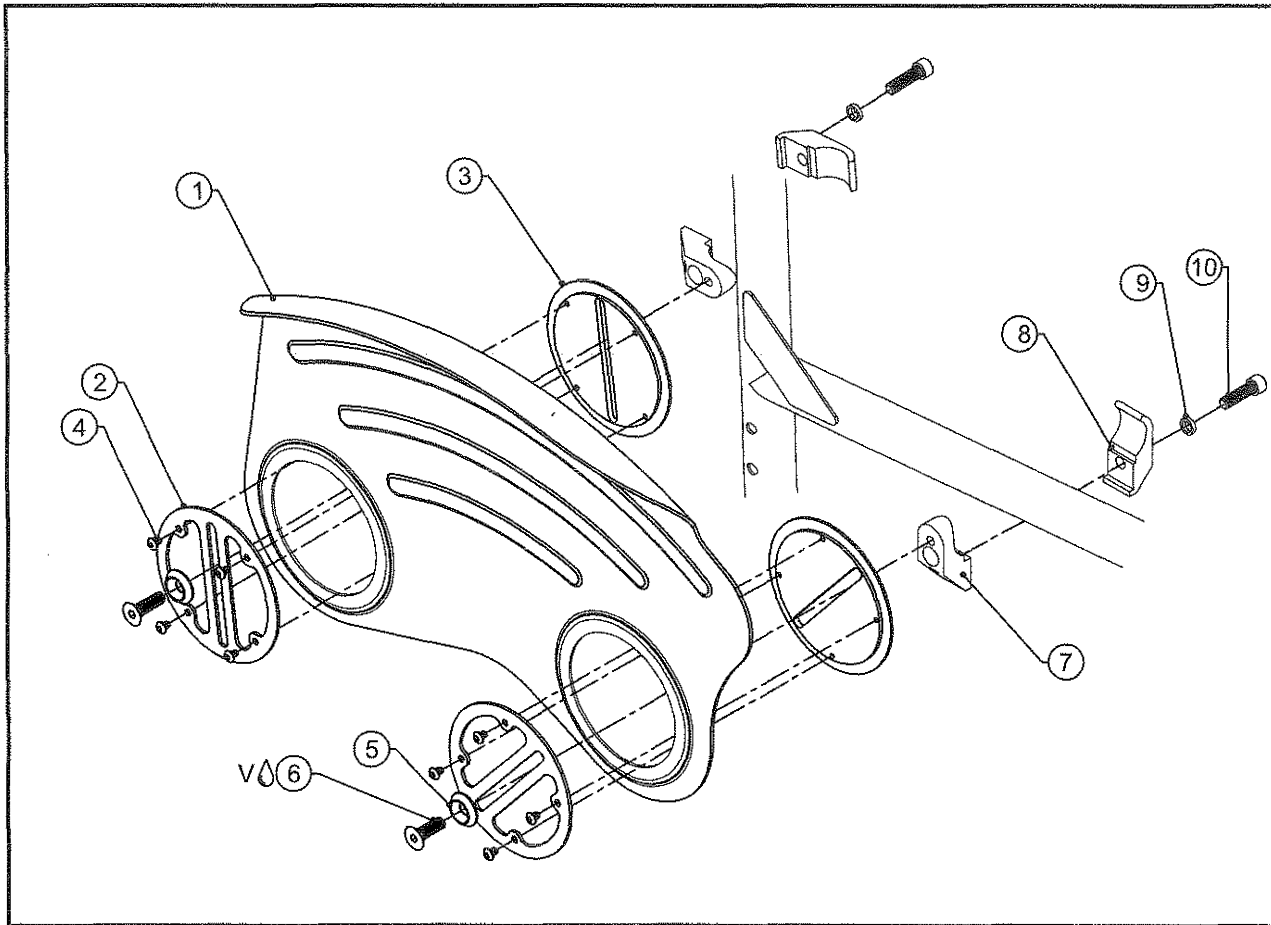


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104353	ADJUSTABLE FENDERED SIDE GUARD, RIGHT
1B	1	CP104354	ADJUSTABLE FENDERED SIDE GUARD, LEFT
1C	1	CP104358	ADJUSTABLE FENDERED SIDE GUARD SHORT, RIGHT
1D	1	CP104359	ADJUSTABLE FENDERED SIDE GUARD SHORT, LEFT
2	2	CP104352	SIDE GUARD CAM COVER
3	2	CP104351	SIDE GUARD CAM
4	8	HD100095	6-32 X 3/16" BUTTON HEAD CAP SCREW, BLACK ZINC W/ THREAD PATCH
5	2	HD100599	1/4" CONICAL WASHER
6△V	2	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
7	2	CP106645	ADJ FENDER SIDE GUARD OUTER CLAMP
8	2	CP106644	ADJ FENDER SIDE GUARD INNER CLAMP
9	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
10	2	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW BLACK ZINC
2-10		AS117967	ADJ. FENDERED SIDE GUARD CAM ASSY, PAIR
1A,2-10		AS117968	ADJ. AL FENDERED SIDE GUARD ASSY, RIGHT
1B,2-10		AS117969	ADJ. AL FENDERED SIDE GUARD ASSY, LEFT
1A,1B,2-10		AS117970	ADJ. AL FENDERED SIDE GUARD ASSY, (PAIR)
1C,2-10		AS117971	ADJ. AL FENDERED SIDE GUARD SHORT ASSY, RIGHT
1D,2-10		AS117972	ADJ. AL FENDERED SIDE GUARD SHORT ASSY, LEFT
1C,1D,2-10		AS117973	ADJ. AL FENDERED SIDE GUARD SHORT ASSY, (PAIR)

TABLE ONE - SEAT HEIGHT MATRIX

SIZE TO USE	REAR SEAT HEIGHT
STANDARD	14.5"-16.5"
SHORT	17"-18.5"

ALUMINUM ADJUSTABLE FENDERED SIDEGUARD ASSEMBLY

TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

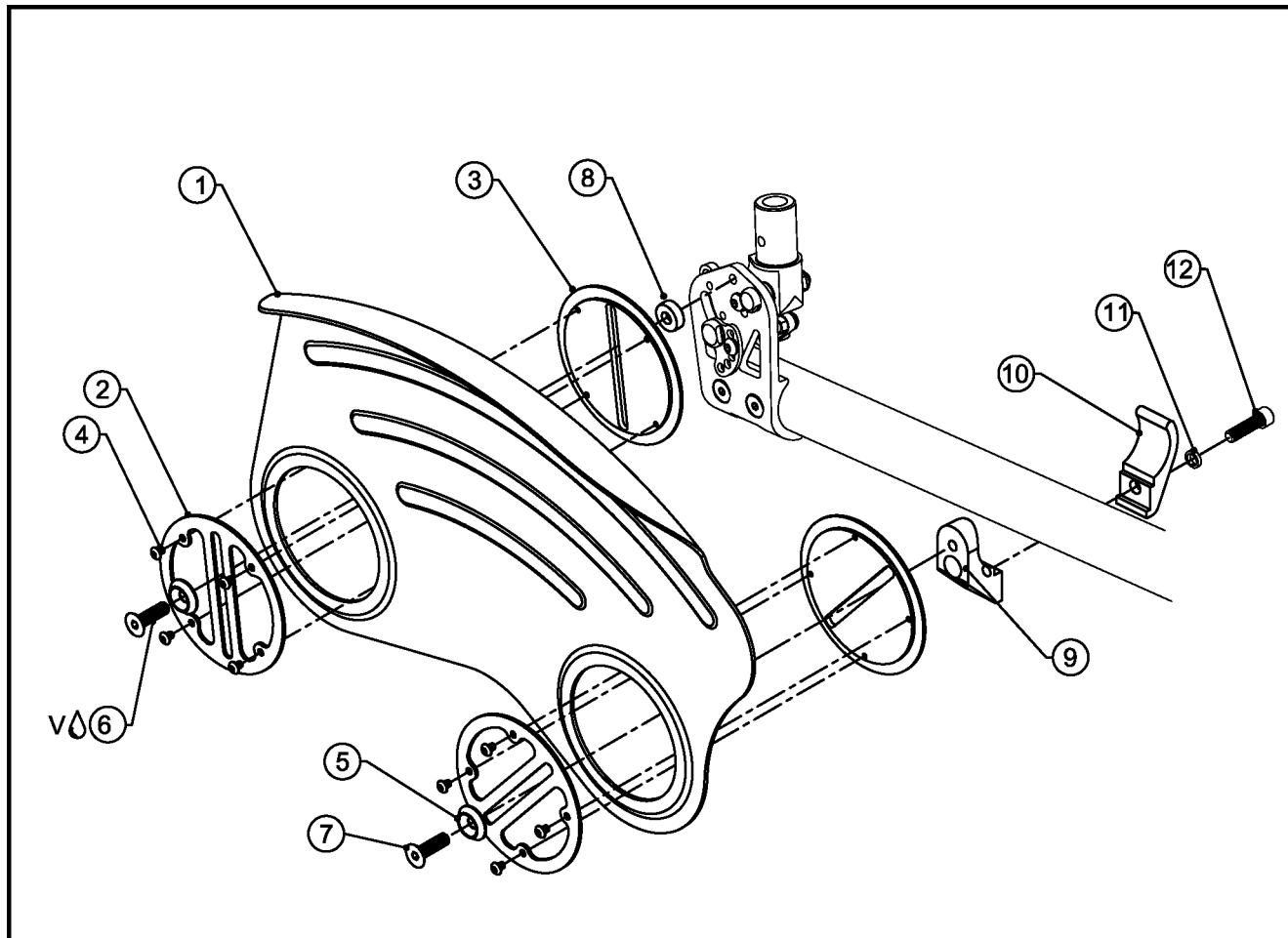


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104353	ADJUSTABLE FENDERED SIDEGUARD, RIGHT
1B	1	CP104354	ADJUSTABLE FENDERED SIDEGUARD, LEFT
1C	1	CP104358	ADJUSTABLE FENDERED SIDEGUARD SHORT, RIGHT
1D	1	CP104359	ADJUSTABLE FENDERED SIDEGUARD SHORT, LEFT
2	2	CP104352	SIDEGUARD CAM COVER
3	2	CP104351	SIDEGUARD CAM
4	8	HD100095	6-32 X 3/16" BUTTON HEAD CAP SCREW, BLACK ZINC WITH THREAD PATCH
5	2	HD100599	1/4" CONICAL WASHER
6V	1	HD100667	1/4-20 X 7/8" FLAT HEAD CAP SCREW BLACK ZINC
7	1	HD100471	1/4-20 X 5/8" FLAT HEAD CAP SCREW BLACK ZINC
8	1	CP104325	SIDEGUARD SPACER
9	1	CP108143	1.25" ADJ FENDER SIDEGUARD CLAMP OUTER
10	1	CP107250	FOLD DOWN SIDE GUARD TOE CLAMP (FOR 1.25" TUBE)
11	1	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
12	1	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW, BLACK ZINC
2-12		AS118173	ADJ. FENDERED SIDEGUARD CAM ASSY, PAIR (Ø1.25 FRAME WITH RAIL)
1A,2-12		AS118174	ADJ. FENDERED SIDEGUARD ASSY 1.25 WITH RAIL, RIGHT
1B,2-12		AS118175	ADJ. FENDERED SIDEGUARD ASSY 1.25 WITH RAIL, LEFT
1A,1B,2-12		AS118176	ADJ. FENDERED SIDEGUARD ASSY 1.25 WITH RAIL (PAIR)
1C,2-12		AS118177	ADJ. FENDERED SIDEGUARD SHORT ASSY 1.25 WITH RAIL, RIGHT
1D,2-12		AS118178	ADJ. FENDERED SIDEGUARD SHORT ASSY 1.25 WITH RAIL, LEFT
1C,1D,2-12		AS118179	ADJ. FENDERED SIDEGUARD SHORT ASSY 1.25 WITH RAIL (PAIR)

TABLE ONE - SEAT HEIGHT MATRIX

SIZE TO USE	REAR SEAT HEIGHT
STANDARD	14.5"-16.5"
SHORT	17"-18.5"

CARBON ADJUSTABLE FENDERED SIDEGUARD ASSEMBLY

TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

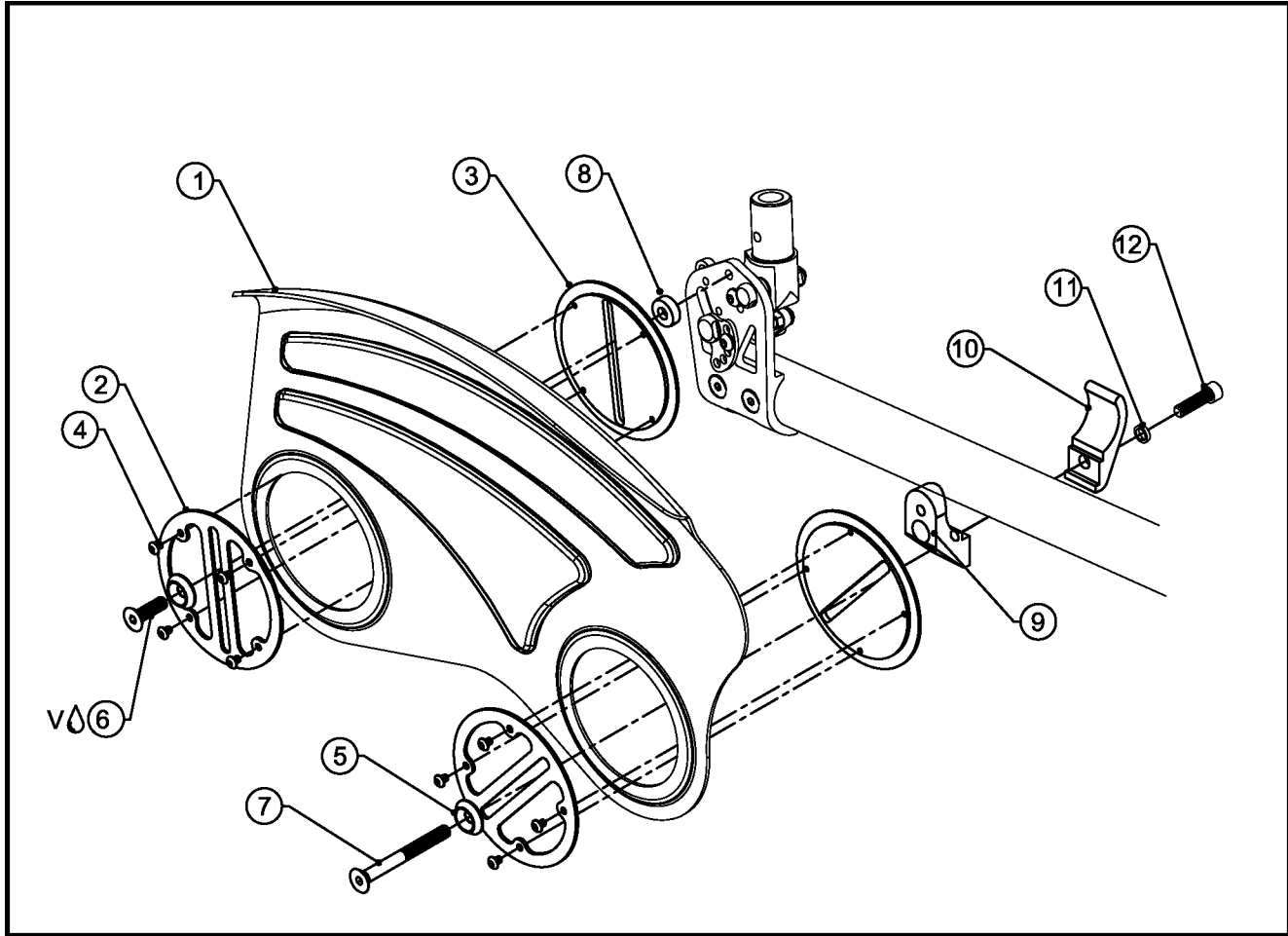


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP107533	ADJUSTABLE CARBON FENDERED SIDEGUARD, RIGHT
1B	1	CP107534	ADJUSTABLE CARBON FENDERED SIDEGUARD, LEFT
1C	1	CP107535	ADJUSTABLE CARBON FENDERED SIDEGUARD SHORT, RIGHT
1D	1	CP107536	ADJUSTABLE CARBON FENDERED SIDEGUARD SHORT, LEFT
2	2	CP104352	SIDEGUARD CAM COVER
3	2	CP104351	SIDEGUARD CAM
4	8	HD100095	6-32 X 3/16" BUTTON HEAD CAP SCREW, BLACK ZINC WITH THREAD PATCH
5	2	HD100599	1/4" CONICAL WASHER
6 ^Δ	1	HD100667	1/4-20 X 7/8" FLAT HEAD CAP SCREW BLACK ZINC
7	1	HD100471	1/4-20 X 5/8" FLAT HEAD CAP SCREW BLACK ZINC
8	1	CP104325	SIDEGUARD SPACER
9	1	CP108143	1.25" ADJ FENDER SIDEGUARD CLAMP OUTER
10	1	CP107250	FOLD DOWN SIDEGUARD TOE CLAMP (FOR 1.25" TUBE)
11	1	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
12	1	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW, BLACK ZINC
2-12		AS118173	ADJ. FENDERED SIDEGUARD CAM ASSY, PAIR (Ø1.25 FRAME WITH RAIL)
1A,2-12		AS118575	ADJ CARBON FENDERED SIDEGUARD ASSY 1.25 WITH RAIL, RIGHT
1B,2-12		AS118576	ADJ. CARBON FENDERED SIDEGUARD ASSY, LEFT
1A,1B,2-12		AS118577	ADJ. CARBON FENDERED SIDEGUARD ASSY (PAIR)
1C,2-12		AS118578	ADJ. CARBON FENDERED SIDEGUARD SHORT ASSY, RIGHT
1D,2-12		AS118579	ADJ. CARBON FENDERED SIDEGUARD SHORT ASSY, LEFT
1C,1D,2-12		AS118580	ADJ. CARBON FENDERED SIDEGUARD SHORT ASSY (PAIR)

TABLE ONE - SEAT HEIGHT MATRIX

SIZE TO USE	REAR SEAT HEIGHT
STANDARD	14.5"-16.5"
SHORT	17"-18.5"

ALUMINUM ADJUSTABLE FENDERED SIDE GUARD ASSEMBLY

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2)

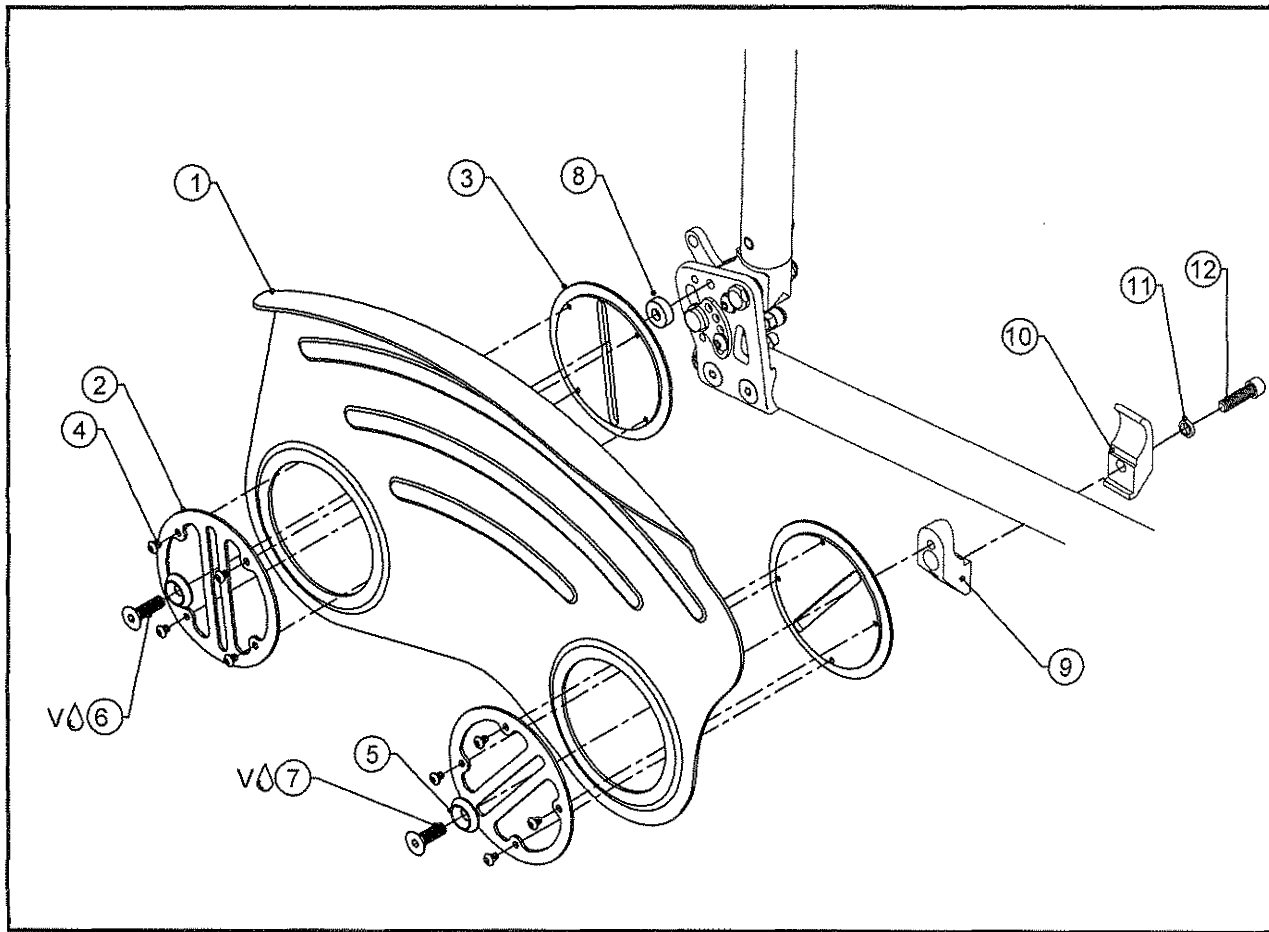

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104353	ADJUSTABLE FENDERED SIDE GUARD, RIGHT
1B	1	CP104354	ADJUSTABLE FENDERED SIDE GUARD, LEFT
1C	1	CP104358	ADJUSTABLE FENDERED SIDE GUARD SHORT, RIGHT
1D	1	CP104359	ADJUSTABLE FENDERED SIDE GUARD SHORT, LEFT
2	2	CP104352	SIDE GUARD CAM COVER
3	2	CP104351	SIDE GUARD CAM
4	8	HD100095	6-32 X 3/16" BUTTON HEAD CAP SCREW, BLACK ZINC W/ THREAD PATCH
5	2	HD100599	1/4" CONICAL WASHER
6V	1	HD100667	1/4-20 X 7/8" FLAT HEAD CAP SCREW BLACK ZINC
7V	1	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
8	1	CP104325	SIDE GUARD SPACER
9	1	CP106645	ADJ FENDER SIDE GUARD OUTER CLAMP
10	1	CP106644	ADJ FENDER SIDE GUARD INNER CLAMP
11	1	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
12	1	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW BLACK ZINC
2-12		AS116069	ADJ. FENDERED SIDEGUARD CAM ASSY, PAIR (Ø1.00 FRAME)
1A,2-12		AS115190	ADJ. FENDERED SIDE GUARD ASSY, 1" FRAME RIGHT
1B,2-12		AS115191	ADJ. FENDERED SIDE GUARD ASSY, 1" FRAME LEFT
1A,1B,2-12		AS115192	ADJ. FENDERED SIDE GUARD ASSY, 1" FRAME (PAIR)
1C,2-12		AS115193	ADJ. FENDERED SIDE GUARD SHORT ASSY, 1" FRAME RIGHT
1D,2-12		AS115194	ADJ. FENDERED SIDE GUARD SHORT ASSY, 1" FRAME LEFT
1C,1D,2-12		AS115195	ADJ. FENDERED SIDE GUARD SHORT ASSY, 1" FRAME (PAIR)


TABLE ONE - SEAT HEIGHT MATRIX

SIZE TO USE	REAR SEAT HEIGHT
STANDARD	14.5"-16.5"
SHORT	17"-18.5"



CARBON ADJUSTABLE FENDERED SIDE GUARD ASSEMBLY

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2)

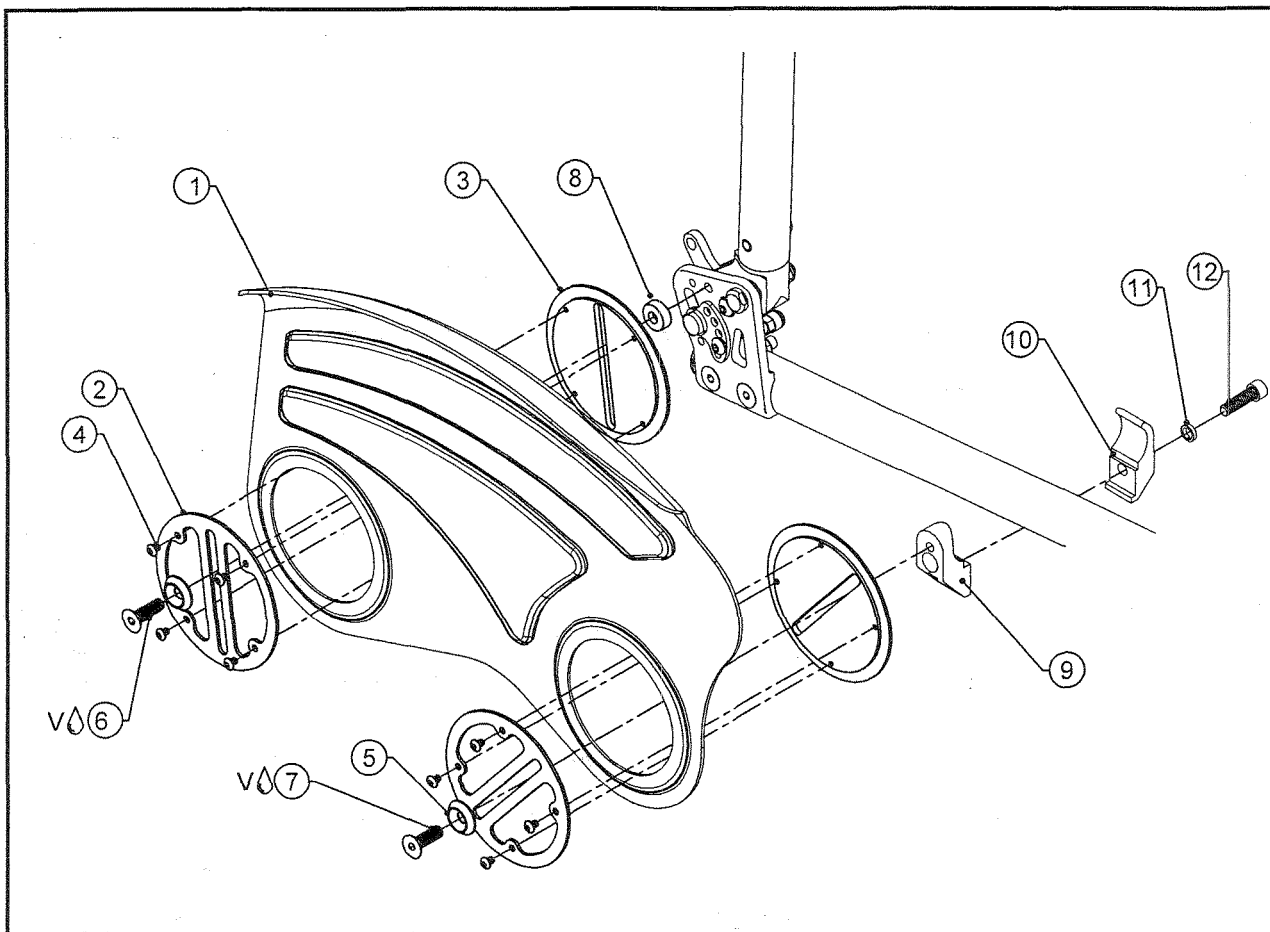


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP107533	ADJUSTABLE CARBON FENDERED SIDE GUARD, RIGHT
1B	1	CP107534	ADJUSTABLE CARBON FENDERED SIDE GUARD, LEFT
1C	1	CP107535	ADJUSTABLE FENDERED SIDE GUARD SHORT, RIGHT
1D	1	CP107536	ADJUSTABLE FENDERED SIDE GUARD SHORT, LEFT
2	2	CP104352	SIDE GUARD CAM COVER
3	2	CP104351	SIDE GUARD CAM
4	8	HD100095	6-32 X 3/16" BUTTON HEAD CAP SCREW, BLACK ZINC W/ THREAD PATCH
5	2	HD100599	1/4" CONICAL WASHER
6 V	1	HD100667	1/4-20 X 7/8" FLAT HEAD CAP SCREW BLACK ZINC
7 V	1	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
8	1	CP104325	SIDE GUARD SPACER
9	1	CP106645	ADJ FENDER SIDE GUARD OUTER CLAMP
10	1	CP106644	ADJ FENDER SIDE GUARD INNER CLAMP
11	1	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
12	1	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW BLACK ZINC
2-12		AS116069	ADJ. FENDERED SIDE GUARD CAM ASSY, PAIR (Ø1.00 FRAME)
1A,2-12		AS117562	ADJ. CARBON FENDERED SIDE GUARD ASSY, 1" FRAME RIGHT
1B,2-12		AS117563	ADJ. CARBON FENDERED SIDE GUARD ASSY, 1" FRAME LEFT
1A,1B,2-12		AS117564	ADJ. CARBON FENDERED SIDE GUARD ASSY, 1" FRAME (PAIR)
1C,2-12		AS117565	ADJ. CARBON FENDERED SIDE GUARD SHORT ASSY, 1" FRAME RIGHT
1D,2-12		AS117566	ADJ. CARBON FENDERED SIDE GUARD SHORT ASSY, 1" FRAME LEFT
1C,1D,2-12		AS117567	ADJ. CARBON FENDERED SIDE GUARD SHORT ASSY, 1" FRAME (PAIR)

TABLE ONE - SEAT HEIGHT MATRIX

SIZE TO USE	REAR SEAT HEIGHT
STANDARD	14.5"-16.5"
SHORT	17"-18.5"

CARBON ADJUSTABLE FENDERED SIDE GUARD ASSEMBLY

TiLite™ AERO X (SERIES 2), 2GX (SERIES 2)

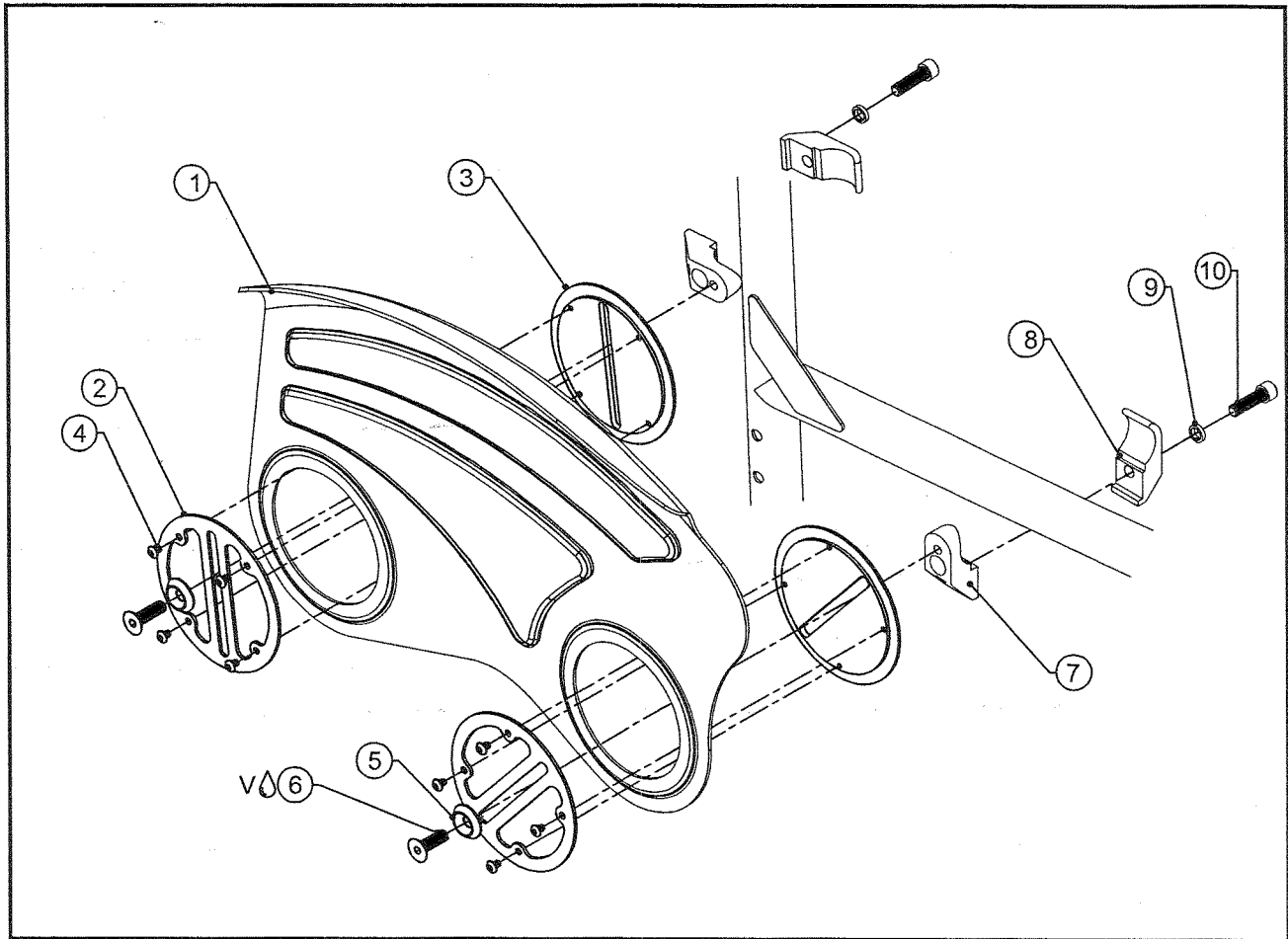


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP107533	ADJUSTABLE CARBON FENDERED SIDE GUARD, RIGHT
1B	1	CP107534	ADJUSTABLE CARBON FENDERED SIDE GUARD, LEFT
1C	1	CP107535	ADJUSTABLE FENDERED SIDE GUARD SHORT, RIGHT
1D	1	CP107536	ADJUSTABLE FENDERED SIDE GUARD SHORT, LEFT
2	2	CP104352	SIDE GUARD CAM COVER
3	2	CP104351	SIDE GUARD CAM
4	8	HD100095	6-32 X 3/16" BUTTON HEAD CAP SCREW, BLACK ZINC W/ THREAD PATCH
5	2	HD100599	1/4" CONICAL WASHER
6V	2	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
7	2	CP106645	ADJ FENDER SIDE GUARD OUTER CLAMP
8	2	CP106644	ADJ FENDER SIDE GUARD INNER CLAMP
9	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
10	2	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW BLACK ZINC
2-10		AS117967	ADJ. FENDERED SIDE GUARD CAM ASSY, PAIR
1A,2-10		AS117961	ADJ. CARBON FENDERED SIDE GUARD ASSY, RIGHT
1B,2-10		AS117962	ADJ. CARBON FENDERED SIDE GUARD ASSY, LEFT
1A,1B,2-10		AS117963	ADJ. CARBON FENDERED SIDE GUARD ASSY, (PAIR)
1C,2-10		AS117964	ADJ. CARBON FENDERED SIDE GUARD SHORT ASSY, RIGHT
1D,2-10		AS117965	ADJ. CARBON FENDERED SIDE GUARD SHORT ASSY, LEFT
1C,1D,2-10		AS117966	ADJ. CARBON FENDERED SIDE GUARD SHORT ASSY, (PAIR)

TABLE ONE - SEAT HEIGHT MATRIX

SIZE TO USE	REAR SEAT HEIGHT
STANDARD	14.5"-16.5"
SHORT	17"-18.5"

ALUMINUM ADJUSTABLE FENDERED SIDEGUARD ASSEMBLY WITH WHEEL LOCK


TiLite™ TWIST

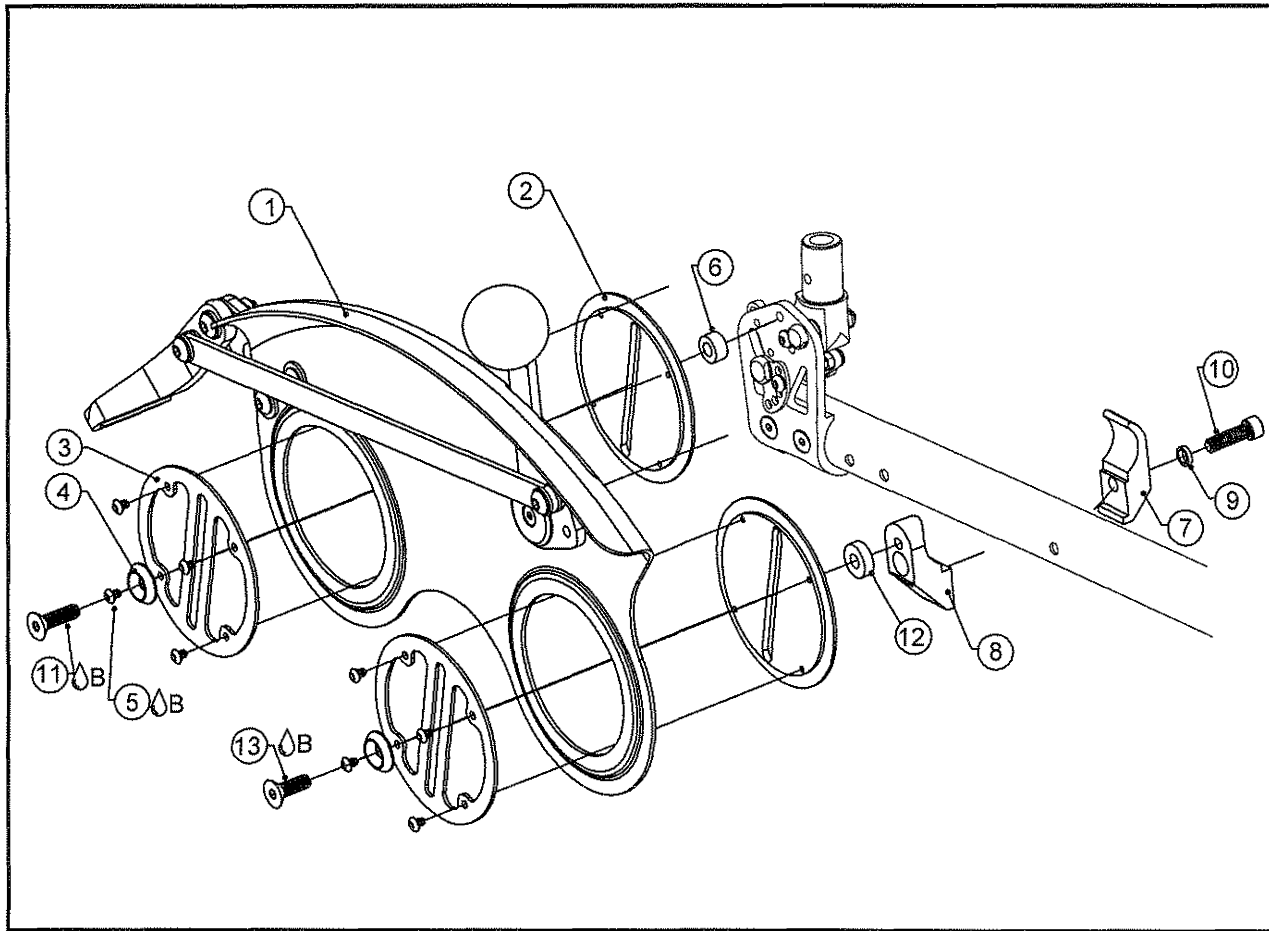

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS115921	ALUMINUM FENDERED SIDEGUARD WITH WHEEL LOCK SUB ASSEMBLY, RIGHT
1B	1	AS115922	ALUMINUM FENDERED SIDEGUARD WITH WHEEL LOCK SUB ASSEMBLY, LEFT
2	2	CP104351	SIDEGUARD CAM
3	2	CP104352	SIDEGUARD CAM COVER
4	2	HD100599	1/4" CONICAL WASHER
5 Δ B	8	HD100095	6-32 X 3/16" BUTTON HEAD CAP SCREW, BLACK ZINC
6	1	CP104325	SIDEGUARD SPACER
7	1	CP106644	ADJ FENDER SIDEGUARD CLAMP INNER
8	1	CP106645	ADJ FENDER SIDEGUARD CLAMP OUTER
9	1	HD100272	1/4 HIGH COLLAR SPLIT LOCK WASHER
10	1	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW, BLACK ZINC
11 Δ B	1	HD100667	1/4-20 X 7/8" FLAT HEAD CAP SCREW, BLACK ZINC
12	1	HD100394	NYLON SPACER
13 Δ B	1	HD100336	1/4-20 X 3/4" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
1A, 2-13	1	AS115923	ALUMINUM FENDERED SIDEGUARD WITH WHEEL LOCK, RIGHT
1B, 2-13	1	AS115924	ALUMINUM FENDERED SIDEGUARD WITH WHEEL LOCK, LEFT

CLOTH SIDE GUARDS


TiLite™ TR, ZR, ZRA, AERO Z, YR, TRA, YRA, Evo, BB, YG, YGS, AERO R, TR (SERIES 2),
YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TWIST,
TR (SERIES 3), AERO T, TRA (SERIES 2)

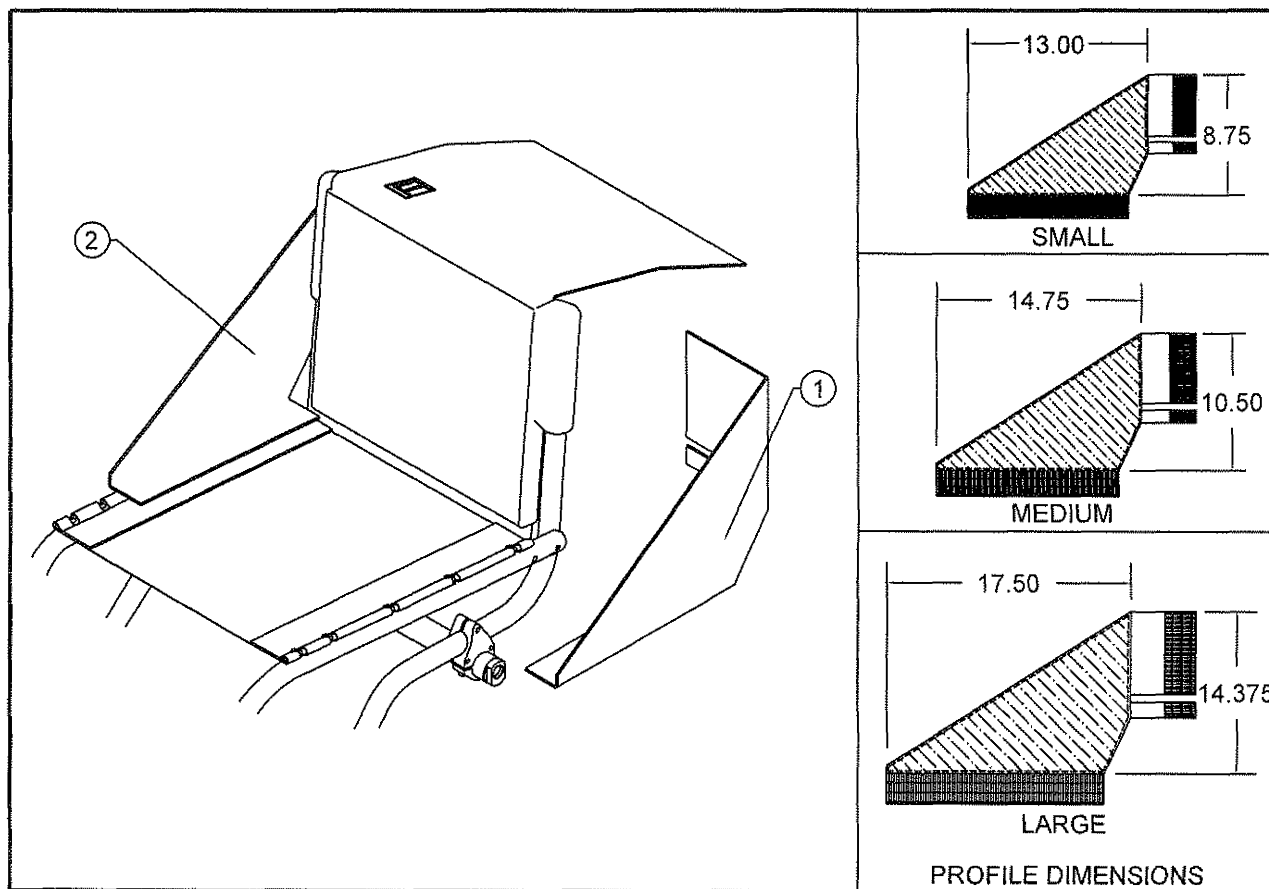

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100089	SIDE GUARD SMALL LT - CLOTH
1B	1	CP100087	SIDE GUARD MEDIUM LT - CLOTH
1C	1	CP101676	SIDE GUARD LARGE LT - CLOTH
2A	1	CP100090	SIDE GUARD SMALL RT - CLOTH
2B	1	CP100088	SIDE GUARD MEDIUM RT - CLOTH
2C	1	CP101677	SIDE GUARD LARGE RT - CLOTH

CLOTH SIDEGUARDS

TiLite™ X, SX, TX, Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

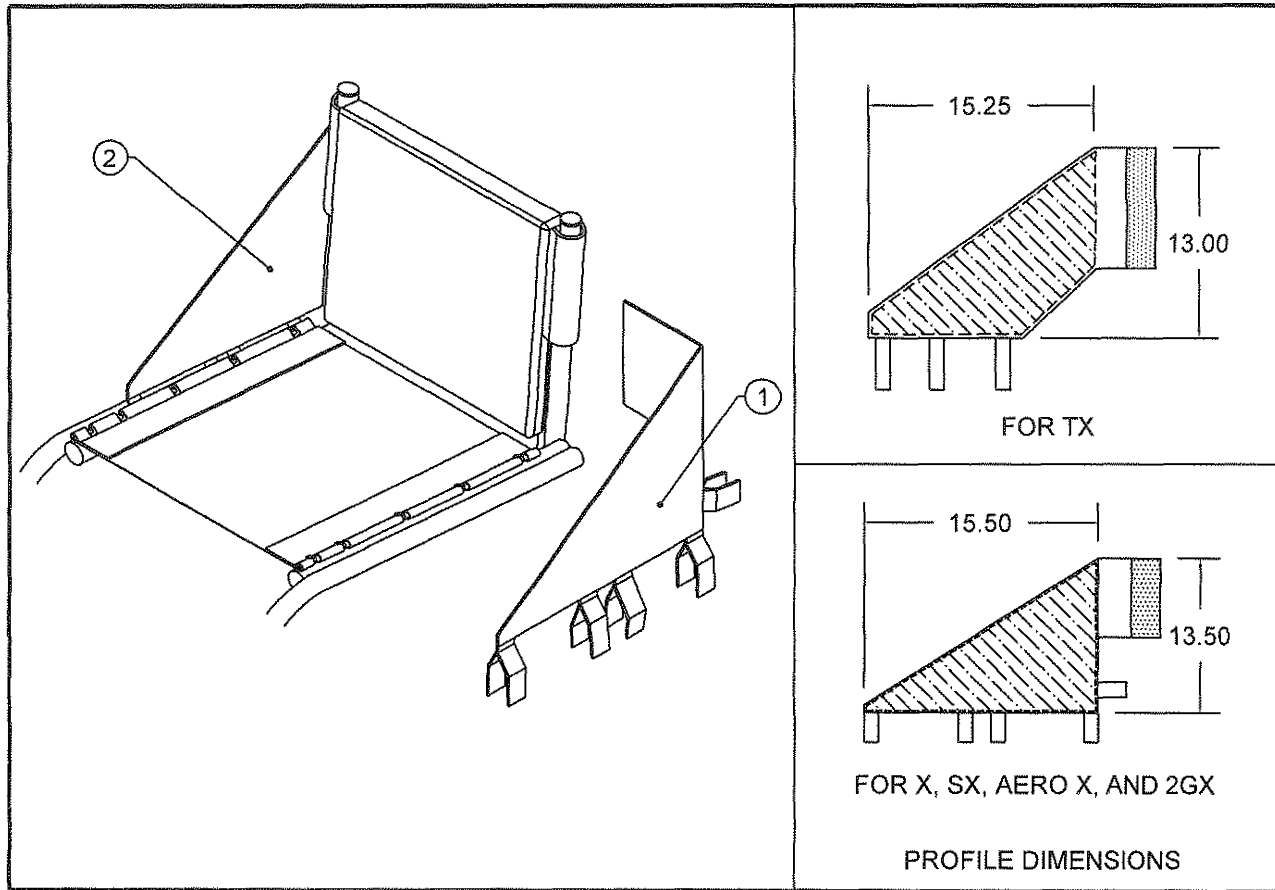


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	SEE TABLE	SIDE GUARD FOLDER LEFT
2	1	SEE TABLE	SIDE GUARD FOLDER RIGHT

CHAIR WIDTH	RIGHT	LEFT
FITS X, SX, AERO X, 2GX	CP100581	CP100580
FITS TX	CP100866	CP100865

ANTI-TIP ASSEMBLY

TiLite™ TR, TRC, YR, TR (SERIES 2), TRC (SERIES 2), YR (SERIES 2), BB

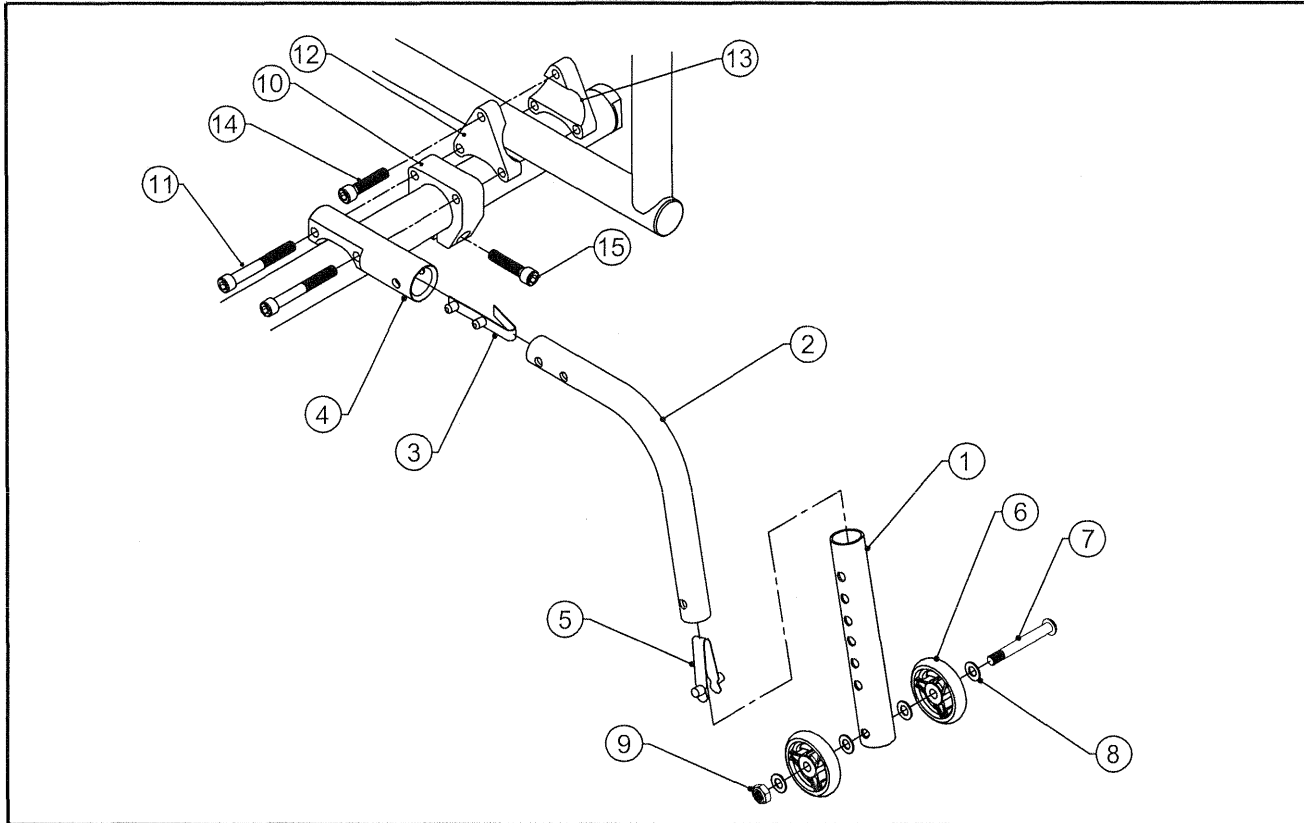


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
2B	1	CP100919	ANTI-TIP EXT TUBE SHORT
2C	1	CP100918	ANTI-TIP EXT TUBE LONG
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4	1	CP101045	ANTI-TIP MOUNT
5	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	1	CP101063	ADJUSTABLE HEIGHT CAMBER TUBE MOUNT
11	2	HD100259	1/4-20 X 3" SOCKET HEAD CAP SCREW
12	1	CP101060	CAMBER MOUNT NON-THREADED
13	1	CP101061	CAMBER MOUNT THREADED
14	1	HD100243	1/4-20 X 1-1/2" SOCKET HEAD CAP SCREW
15	1	HD100051	1/4-20 X 1" SOCKET HEAD CAP SCREW
2A,3,5		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
2B,3,5		AS100570	ANTI-TIP TUBE, SHORT ASSEMBLY
2C,3,5		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,6-9		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,6-9		AS103462	ANTI-TIP EXTENSION ASSY, LONG
1A,2A,3-15		AS104499	EX SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY
1A,2A,3-15		AS104500	EX SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B,2A,3-15		AS104501	EX SHORT ANTI-TIP W/ LONG EXT ASSEMBLY
1B,2A,3-15		AS104502	EX SHORT ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)
1A,2B,3-15		AS104436	SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY
1A,2B,3-15		AS100961	SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B,2B,3-15		AS104437	SHORT ANTI-TIP W/ LONG EXT ASSEMBLY
1B,2B,3-15		AS100962	SHORT ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)
1A,2C,3-15		AS104438	LONG ANTI-TIP W/ SHORT EXT ASSEMBLY
1A,2C,3-15		AS100967	LONG ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B,2C,3-15		AS104439	LONG ANTI-TIP W/ LONG EXT ASSEMBLY
1B,2C,3-15		AS100968	LONG ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)



ANTI-TIP ASSEMBLY

TiLite™ ZR

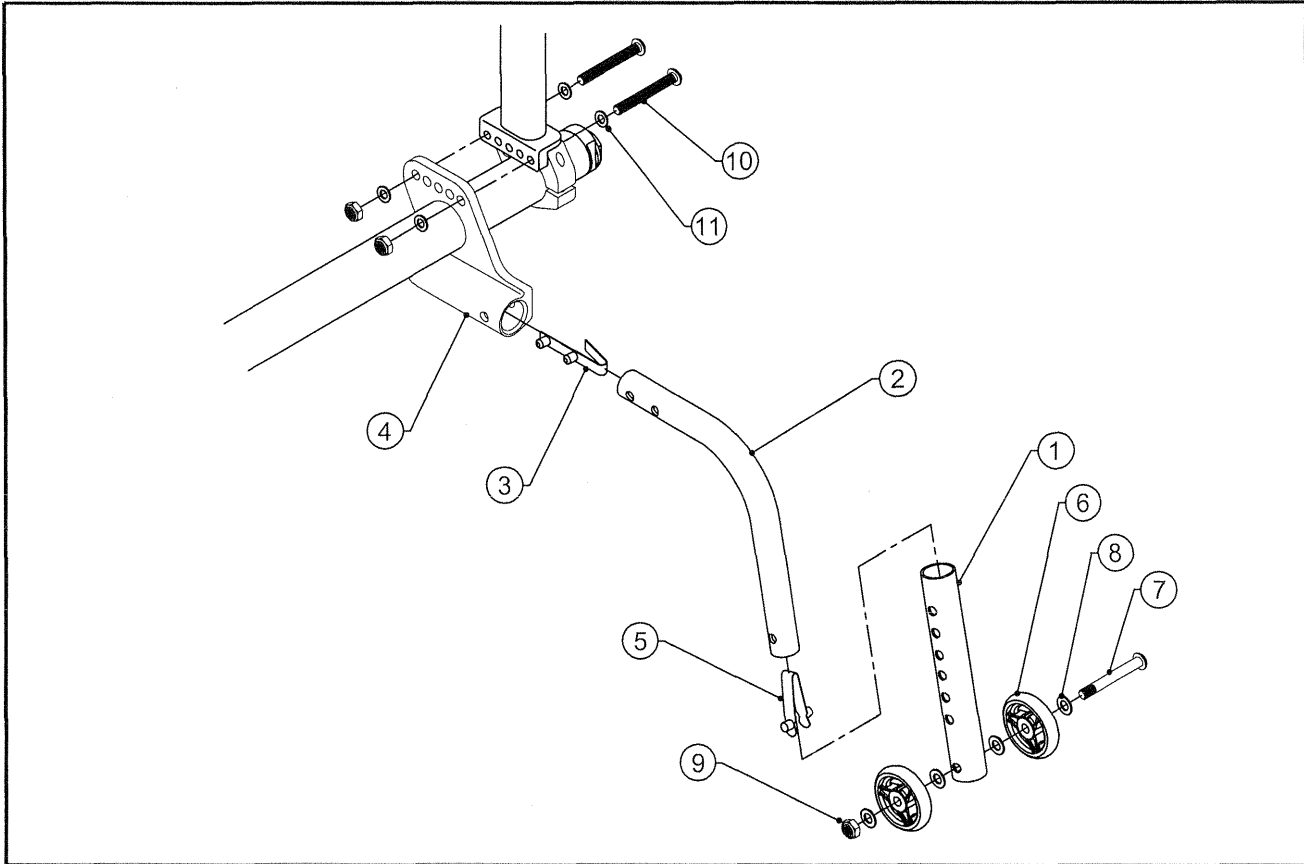

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
2B	1	CP101312	ANTI-TIP EXT TUBE SHORT
2C	1	CP100919	ANTI-TIP EXT TUBE MEDIUM
2D	1	CP100918	ANTI-TIP EXT TUBE LONG
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4A	1	CP103180	ANTI-TIP RECEPTICAL RIGHT
4B	1	CP103181	ANTI-TIP RECEPTICAL LEFT
5	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	3	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100324	1/4-28 X 1-3/4 BUTTON HEAD CAP SCREW FULL THREAD, BLACK ZINC
11	4	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
2A,3,5		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
2B,3,5		AS101047	ANTI-TIP TUBE, SHORT ASSEMBLY
2C,3,5		AS100570	ANTI-TIP TUBE, MEDIUM ASSEMBLY
2D,3,5		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,6-9		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,6-9		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE

ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ ZR

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A-4A,5-11		AS104526	EXTRA SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2A-4B,5-11		AS104527	EXTRA SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2A,3-11		AS104524	EXTRA SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2A-4A,5-11		AS104528	EXTRA SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2A-4B,5-11		AS104529	EXTRA SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2A,3-11		AS104525	EXTRA SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2B-4A,5-11		AS104530	SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2B-4B,5-11		AS104531	SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2B,3-11		AS104193	SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2B-4A,5-11		AS104532	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B-4B,5-11		AS104533	SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2B,3-11		AS104194	SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2C-4A,5-11		AS104534	MEDIUM ANTI-TIP W/ SHORT EXT, RIGHT
1A,2C-4B,5-11		AS104535	MEDIUM ANTI-TIP W/ SHORT EXT, LEFT
1A,2C,3-11		AS104195	MEDIUM ANTI-TIP W/ SHORT EXT (PAIR)
1B,2C-4A,5-11		AS104536	MEDIUM ANTI-TIP W/ LONG EXT, RIGHT
1B,2C-4B,5-11		AS104537	MEDIUM ANTI-TIP W/ LONG EXT, LEFT
1B,2C,3-11		AS104196	MEDIUM ANTI-TIP W/ LONG EXT (PAIR)
1A,2D-4A,5-11		AS104538	LONG ANTI-TIP W/ SHORT EXT, RIGHT
1A,2D-4B,5-11		AS104539	LONG ANTI-TIP W/ SHORT EXT, LEFT
1A,2D,3-11		AS104197	LONG ANTI-TIP W/ SHORT EXT (PAIR)
1B,2D-4A,5-11		AS104540	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2D-4B,5-11		AS104541	LONG ANTI-TIP W/ LONG EXT, LEFT
1B,2D,3-11		AS104198	LONG ANTI-TIP W/ LONG EXT (PAIR)



ANTI-TIP ASSEMBLY

TiLite™ ZR (SERIES 2)

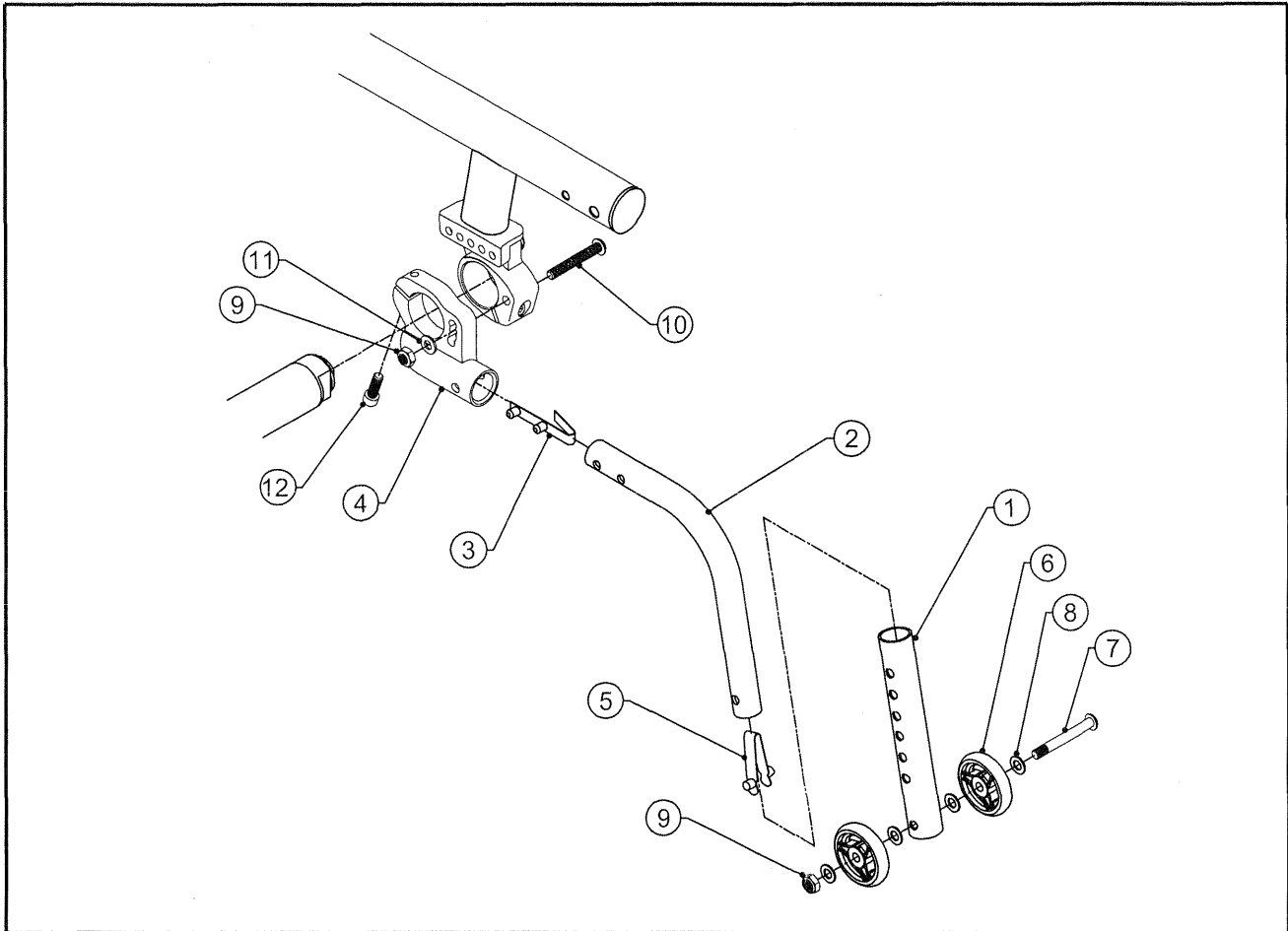


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
2B	1	CP101312	ANTI-TIP EXT TUBE SHORT
2C	1	CP100919	ANTI-TIP EXT TUBE MEDIUM
2D	1	CP100918	ANTI-TIP EXT TUBE LONG
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4A	1	CP104600	ANTI-TIP RECEPTICAL, RIGHT
4B	1	CP104601	ANTI-TIP RECEPTICAL, LEFT
5	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	1	HD100324	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW FULL THREAD, BLACK ZINC
11	1	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
12	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
2A,3,5		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
2B,3,5		AS101047	ANTI-TIP TUBE, SHORT ASSEMBLY
2C,3,5		AS100570	ANTI-TIP TUBE, MEDIUM ASSEMBLY
2D,3,5		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,6-9		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,6-9		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE

ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ ZR (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A-4A,5-12		AS110753	EXTRA SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2A-4B,5-12		AS110754	EXTRA SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2A,3-12		AS110755	EXTRA SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2A-4A,5-12		AS110756	EXTRA SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2A-4B,5-12		AS110757	EXTRA SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2A,3-12		AS110758	EXTRA SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2B-4A,5-12		AS110759	SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2B-4B,5-12		AS110760	SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2B,3-12		AS110761	SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2B-4A,5-12		AS110762	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B-4B,5-12		AS110763	SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2B,3-12		AS110764	SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2C-4A,5-12		AS110765	MEDIUM ANTI-TIP W/ SHORT EXT, RIGHT
1A,2C-4B,5-12		AS110766	MEDIUM ANTI-TIP W/ SHORT EXT, LEFT
1A,2C,3-12		AS110767	MEDIUM ANTI-TIP W/ SHORT EXT (PAIR)
1B,2C-4A,5-12		AS110768	MEDIUM ANTI-TIP W/ LONG EXT, RIGHT
1B,2C-4B,5-12		AS110769	MEDIUM ANTI-TIP W/ LONG EXT, LEFT
1B,2C,3-12		AS110770	MEDIUM ANTI-TIP W/ LONG EXT (PAIR)
1A,2D-4A,5-12		AS110771	LONG ANTI-TIP W/ SHORT EXT, RIGHT
1A,2D-4B,5-12		AS110772	LONG ANTI-TIP W/ SHORT EXT, LEFT
1A,2D,3-12		AS110773	LONG ANTI-TIP W/ SHORT EXT (PAIR)
1B,2D-4A,5-12		AS110774	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2D-4B,5-12		AS110775	LONG ANTI-TIP W/ LONG EXT, LEFT
1B,2D,3-12		AS110776	LONG ANTI-TIP W/ LONG EXT (PAIR)

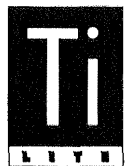

ANTI-TIP ASSEMBLY

TiLite™ ZRA, AERO Z

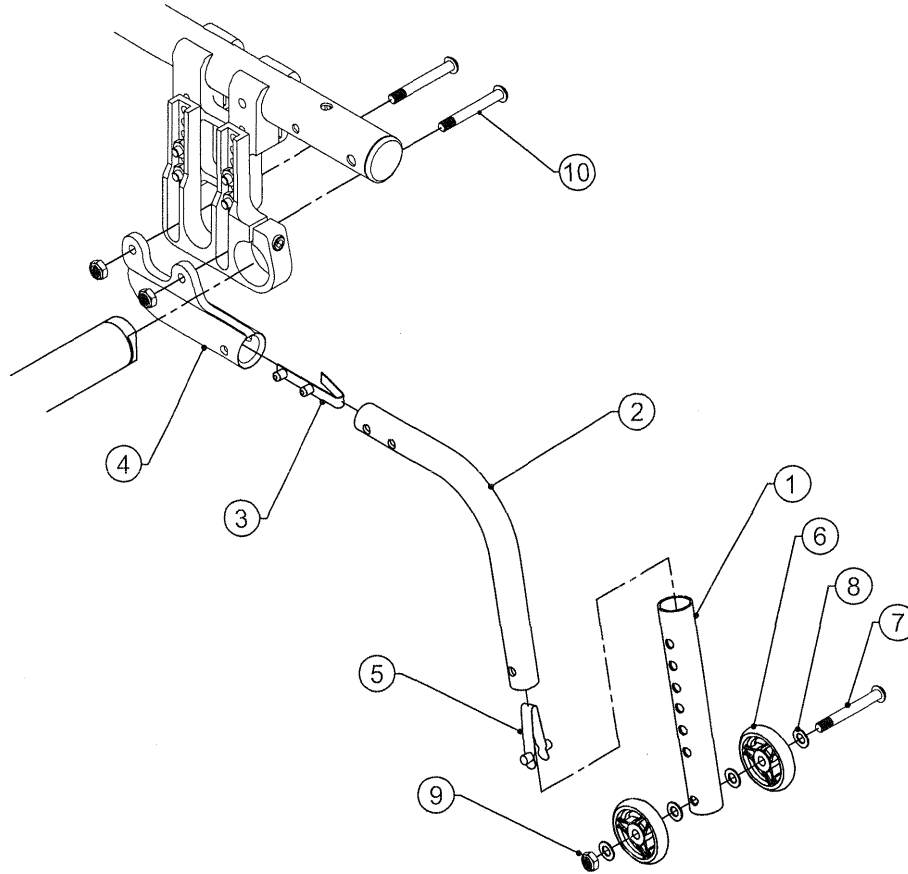

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
2B	1	CP101312	ANTI-TIP EXT TUBE SHORT
2C	1	CP100919	ANTI-TIP EXT TUBE MEDIUM
2D	1	CP100918	ANTI-TIP EXT TUBE LONG
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4	1	CP102067	ANTI-TIP RECEPTICAL
5	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	3	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100342	1/4-28 X 1 1/4" BUTTON HEAD CAP SCREW, BLACK ZINC
2A,3,5		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
2B,3,5		AS101047	ANTI-TIP TUBE, SHORT ASSEMBLY
2C,3,5		AS100570	ANTI-TIP TUBE, MEDIUM ASSEMBLY
2D,3,5		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,6-9		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,6-9		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE

ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ ZRA, AERO Z

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3-10		AS104516	EXTRA SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY
1A,2A,3-10		AS104517	EXTRA SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B,2A,3-10		AS104518	EXTRA SHORT ANTI-TIP W/ LONG EXT ASSEMBLY
1B,2A,3-10		AS104519	EXTRA SHORT ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)
1A,2B,3-10		AS104448	SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY
1A,2B,3-10		AS102367	SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B,2B,3-10		AS104449	SHORT ANTI-TIP W/ LONG EXT ASSEMBLY
1B,2B,3-10		AS102368	SHORT ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)
1A,2C,3-10		AS104450	MEDIUM ANTI-TIP W/ SHORT EXT ASSEMBLY
1A,2C,3-10		AS102369	MEDIUM ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B,2C,3-10		AS104451	MEDIUM ANTI-TIP W/ LONG EXT ASSEMBLY
1B,2C,3-10		AS102370	MEDIUM ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)
1A,2D,3-10		AS104452	LONG ANTI-TIP W/ SHORT EXT ASSEMBLY
1A,2D,3-10		AS102371	LONG ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B,2D,3-10		AS104453	LONG ANTI-TIP W/ LONG EXT ASSEMBLY
1B,2D,3-10		AS102372	LONG ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)



ANTI-TIP ASSEMBLY

TiLite™ ZRA (SERIES 2), AERO Z (SERIES 2), TWIST, AERO T, TRA (SERIES 2)

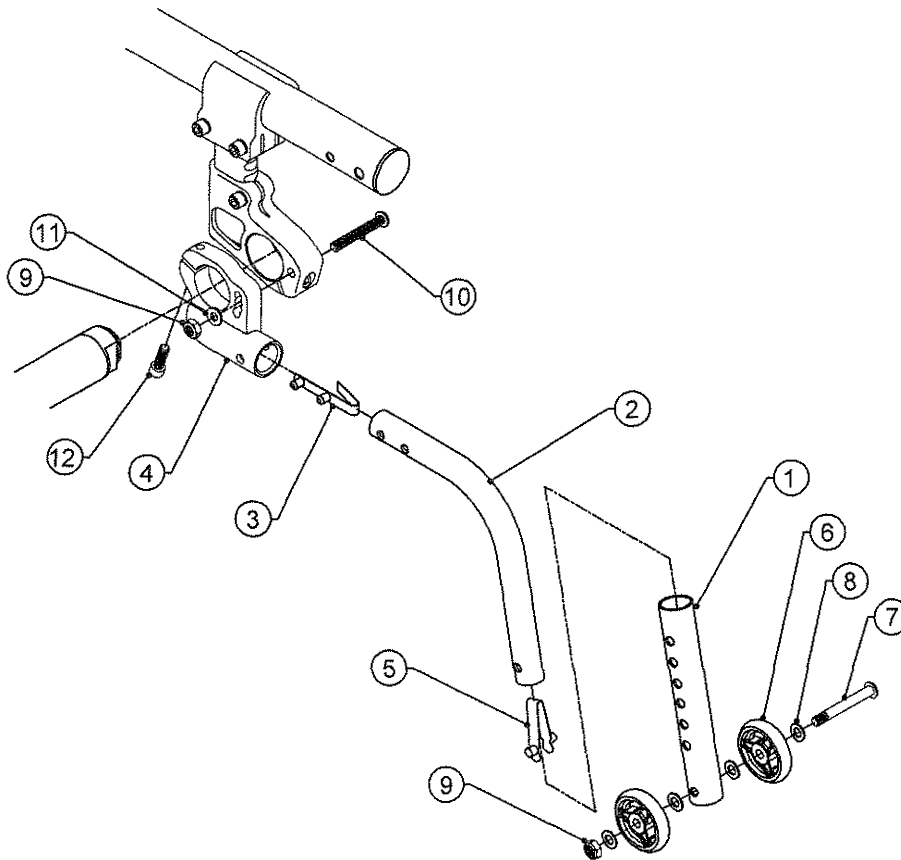

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
2B	1	CP101312	ANTI-TIP EXT TUBE SHORT
2C	1	CP100919	ANTI-TIP EXT TUBE MEDIUM
2D	1	CP100918	ANTI-TIP EXT TUBE LONG
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4A	1	CP104600	ANTI-TIP RECEPTICAL, RIGHT
4B	1	CP104601	ANTI-TIP RECEPTICAL, LEFT
5	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	1	HD100324	1/4-28 X 1-3/4 BUTTON HEAD CAP SCREW FULL THREAD, BLACK ZINC
11	1	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
12	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
2A,3,5		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
2B,3,5		AS101047	ANTI-TIP TUBE, SHORT ASSEMBLY
2C,3,5		AS100570	ANTI-TIP TUBE, MEDIUM ASSEMBLY
2D,3,5		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,6-9		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,6-9		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE



Parts Manual

ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ ZRA (SERIES 2), AERO Z (SERIES 2), TWIST, AERO T, TRA (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A-4A,5-12		AS110753	EXTRA SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2A-4B,5-12		AS110754	EXTRA SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2A,3-12		AS110755	EXTRA SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2A-4A,5-12		AS110756	EXTRA SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2A-4B,5-12		AS110757	EXTRA SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2A,3-12		AS110758	EXTRA SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2B-4A,5-12		AS110759	SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2B-4B,5-12		AS110760	SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2B,3-12		AS110761	SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2B-4A,5-12		AS110762	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B-4B,5-12		AS110763	SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2B,3-12		AS110764	SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2C-4A,5-12		AS110765	MEDIUM ANTI-TIP W/ SHORT EXT, RIGHT
1A,2C-4B,5-12		AS110766	MEDIUM ANTI-TIP W/ SHORT EXT, LEFT
1A,2C,3-12		AS110767	MEDIUM ANTI-TIP W/ SHORT EXT (PAIR)
1B,2C-4A,5-12		AS110768	MEDIUM ANTI-TIP W/ LONG EXT, RIGHT
1B,2C-4B,5-12		AS110769	MEDIUM ANTI-TIP W/ LONG EXT, LEFT
1B,2C,3-12		AS110770	MEDIUM ANTI-TIP W/ LONG EXT (PAIR)
1A,2D-4A,5-12		AS110771	LONG ANTI-TIP W/ SHORT EXT, RIGHT
1A,2D-4B,5-12		AS110772	LONG ANTI-TIP W/ SHORT EXT, LEFT
1A,2D,3-12		AS110773	LONG ANTI-TIP W/ SHORT EXT (PAIR)
1B,2D-4A,5-12		AS110774	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2D-4B,5-12		AS110775	LONG ANTI-TIP W/ LONG EXT, LEFT
1B,2D,3-12		AS110776	LONG ANTI-TIP W/ LONG EXT (PAIR)



ANTI-TIP ASSEMBLY

TiLite™ TR (SERIES 3)

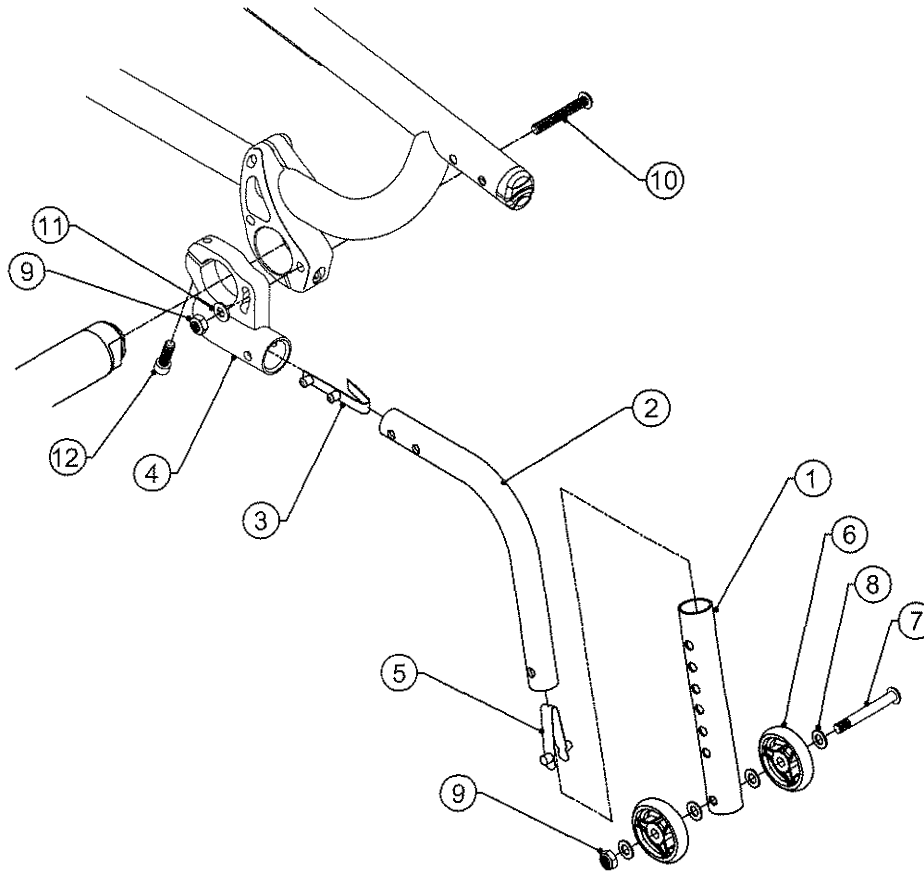

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
2B	1	CP101312	ANTI-TIP EXT TUBE SHORT
2C	1	CP100919	ANTI-TIP EXT TUBE MEDIUM
2D	1	CP100918	ANTI-TIP EXT TUBE LONG
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4A	1	CP104600	ANTI-TIP RECEPTICAL, RIGHT
4B	1	CP104601	ANTI-TIP RECEPTICAL, LEFT
5	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	1	HD100324	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW FULL THREAD, BLACK ZINC
11	1	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
12	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
2A,3,5		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
2B,3,5		AS101047	ANTI-TIP TUBE, SHORT ASSEMBLY
2C,3,5		AS100570	ANTI-TIP TUBE, MEDIUM ASSEMBLY
2D,3,5		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,6-9		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,6-9		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE



Parts Manual

ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ TR (SERIES 3)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A-4A,5-12		AS110753	EXTRA SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2A-4B,5-12		AS110754	EXTRA SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2A,3-12		AS110755	EXTRA SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2A-4A,5-12		AS110756	EXTRA SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2A-4B,5-12		AS110757	EXTRA SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2A,3-12		AS110758	EXTRA SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2B-4A,5-12		AS110759	SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1A,2B-4B,5-12		AS110760	SHORT ANTI-TIP W/ SHORT EXT, LEFT
1A,2B,3-12		AS110761	SHORT ANTI-TIP W/ SHORT EXT (PAIR)
1B,2B-4A,5-12		AS110762	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B-4B,5-12		AS110763	SHORT ANTI-TIP W/ LONG EXT, LEFT
1B,2B,3-12		AS110764	SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2C-4A,5-12		AS110765	MEDIUM ANTI-TIP W/ SHORT EXT, RIGHT
1A,2C-4B,5-12		AS110766	MEDIUM ANTI-TIP W/ SHORT EXT, LEFT
1A,2C,3-12		AS110767	MEDIUM ANTI-TIP W/ SHORT EXT (PAIR)
1B,2C-4A,5-12		AS110768	MEDIUM ANTI-TIP W/ LONG EXT, RIGHT
1B,2C-4B,5-12		AS110769	MEDIUM ANTI-TIP W/ LONG EXT, LEFT
1B,2C,3-12		AS110770	MEDIUM ANTI-TIP W/ LONG EXT (PAIR)
1A,2D-4A,5-12		AS110771	LONG ANTI-TIP W/ SHORT EXT, RIGHT
1A,2D-4B,5-12		AS110772	LONG ANTI-TIP W/ SHORT EXT, LEFT
1A,2D,3-12		AS110773	LONG ANTI-TIP W/ SHORT EXT (PAIR)
1B,2D-4A,5-12		AS110774	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2D-4B,5-12		AS110775	LONG ANTI-TIP W/ LONG EXT, LEFT
1B,2D,3-12		AS110776	LONG ANTI-TIP W/ LONG EXT (PAIR)



ANTI-TIP ASSEMBLY


TiLite™ TRA, YRA

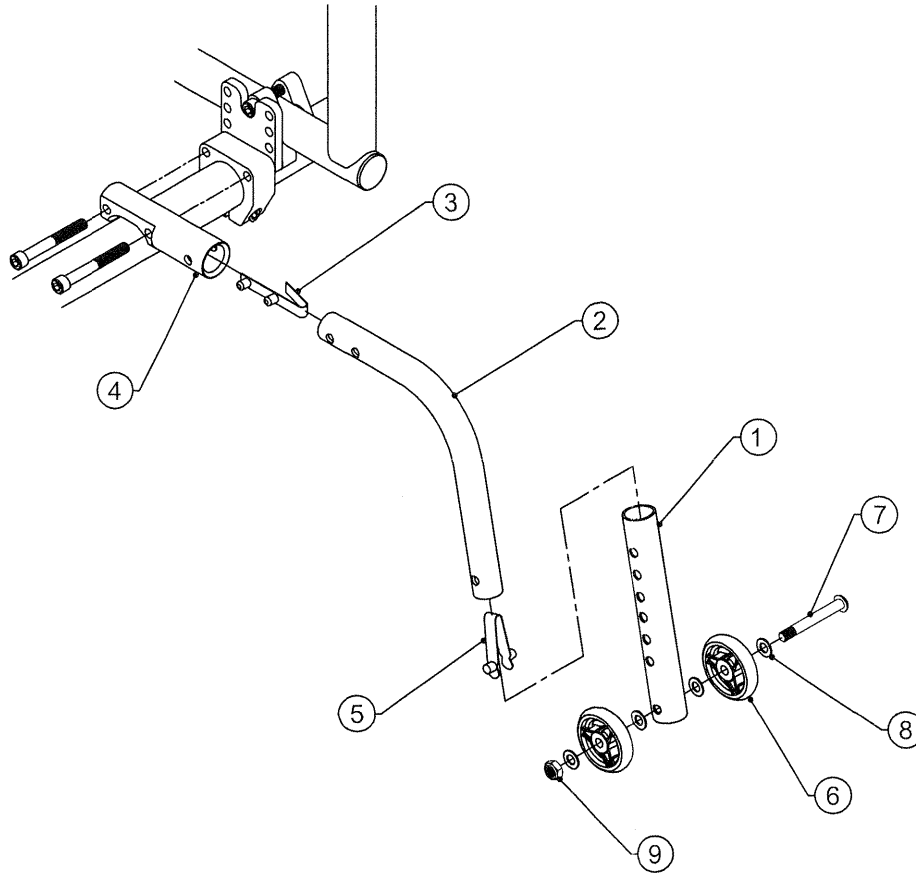

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP100919	ANTI-TIP EXT TUBE SHORT
2B	1	CP100918	ANTI-TIP EXT TUBE LONG
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4	1	CP101045	ANTI-TIP MOUNT
5	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
2A, 3, 5		AS100570	ANTI-TIP TUBE, SHORT ASSEMBLY
2B, 3, 5		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A, 6-9		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B, 6-9		AS103462	ANTI-TIP EXTENSION ASSY, LONG
1A, 2A, 3-9		AS104432	SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY
1A, 2A, 3-9		AS100941	SHORT ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B, 2A, 3-9		AS104433	SHORT ANTI-TIP W/ LONG EXT ASSEMBLY
1B, 2A, 3-9		AS100920	SHORT ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)
1A, 2B, 3-9		AS104434	LONG ANTI-TIP W/ SHORT EXT ASSEMBLY
1A, 2B, 3-9		AS100921	LONG ANTI-TIP W/ SHORT EXT ASSEMBLY (PAIR)
1B, 2B, 3-9		AS104435	LONG ANTI-TIP W/ LONG EXT ASSEMBLY
1B, 2B, 3-9		AS100922	LONG ANTI-TIP W/ LONG EXT ASSEMBLY (PAIR)

ANTI-TIP ASSEMBLY


TiLite™ EVO

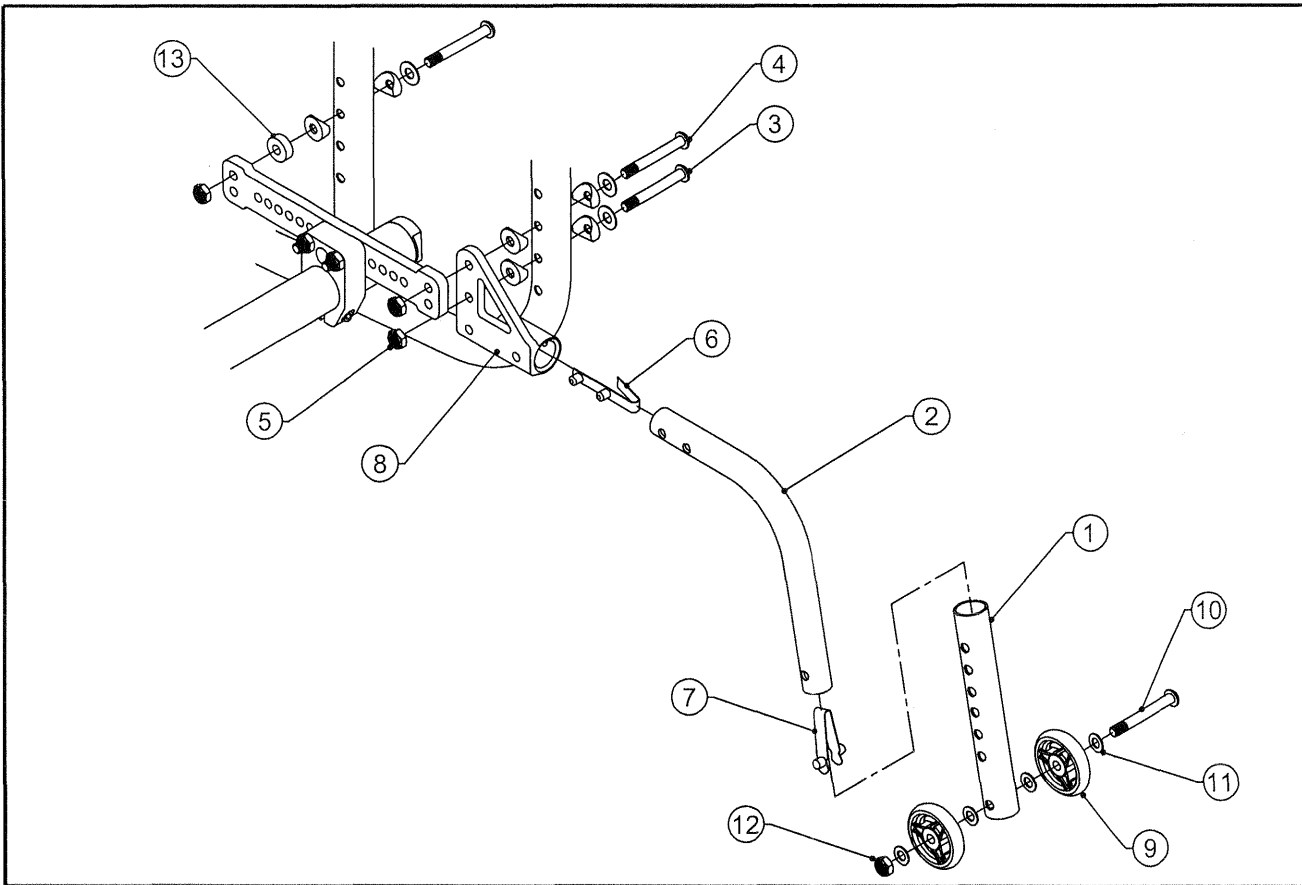

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP102036	SHORT DEPTH ANTI-TIP EXTENTION SHORT
2B	1	CP102037	SHORT DEPTH ANTI-TIP EXTENTION LONG
3	1	CP100241	1/4-28 X 1-5/8" BUTTON HEAD CAP SCREW
4	2	HD100105	1/4-28 X 2-1/4" BUTTON HEAD CAP SCREW
5	3	HD100021	1/4-28 X .218 NYLOCK LOCKNUT
6	1	HD100232	SINGLE END DUAL SNAP BUTTON
7	1	HD100155	SINGLE END SNAP BUTTON
8A	1	CP100916	ANTI-TIP MOUNT LEFT
8B	1	CP100917	ANTI-TIP MOUNT RIGHT
9	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
10	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
11	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
12	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
13	1	CP102035	NYLON SPACER .75 O.D. X .25 I.D. X .25 TH
2A,6-7		AS100570	ANTI-TIP TUBE, SHORT ASSEMBLY
2B,6-7		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,9-12		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,9-12		AS103462	ANTI-TIP EXTENSION ASSY, LONG
1A,2A,3-8A,9-13		AS104479	ANTI-TIP SHORT/SHORT EXT ASSEMBLY, LEFT
1A,2A,3-8B,9-13		AS104480	ANTI-TIP SHORT/SHORT EXT ASSEMBLY, RIGHT
1A,2A,3-13		AS102185	ANTI-TIP SHORT/SHORT EXT ASSEMBLY (PAIR)
1A,2B,3-8A,9-13		AS104481	ANTI-TIP LONG/SHORT EXT ASSEMBLY, LEFT
1A,2B,3-8B,9-13		AS104482	ANTI-TIP LONG/SHORT EXT ASSEMBLY, RIGHT
1A,2B,3-13		AS102186	ANTI-TIP LONG/SHORT EXT ASSEMBLY (PAIR)
1B,2A,3-8A,9-13		AS104483	ANTI-TIP SHORT/LONG EXT ASSEMBLY, LEFT
1B,2A,3-8B,9-13		AS104484	ANTI-TIP SHORT/LONG EXT ASSEMBLY, RIGHT
1B,2A,3-13		AS102187	ANTI-TIP SHORT/LONG EXT ASSEMBLY (PAIR)
1B,2B,3-8A,9-13		AS104485	ANTI-TIP LONG/LONG EXT ASSEMBLY, LEFT
1B,2B,3-8B,9-13		AS104486	ANTI-TIP LONG/LONG EXT ASSEMBLY, RIGHT
1B,2B,3-13		AS102188	ANTI-TIP LONG/LONG EXT ASSEMBLY (PAIR)

ANTI-TIP ASSEMBLY FOR FROG LEG® REAR SUSPENSION


TiLite™ TR(SERIES 2), TRC(SERIES 2), YR(SERIES 2), TR (SERIES 3)

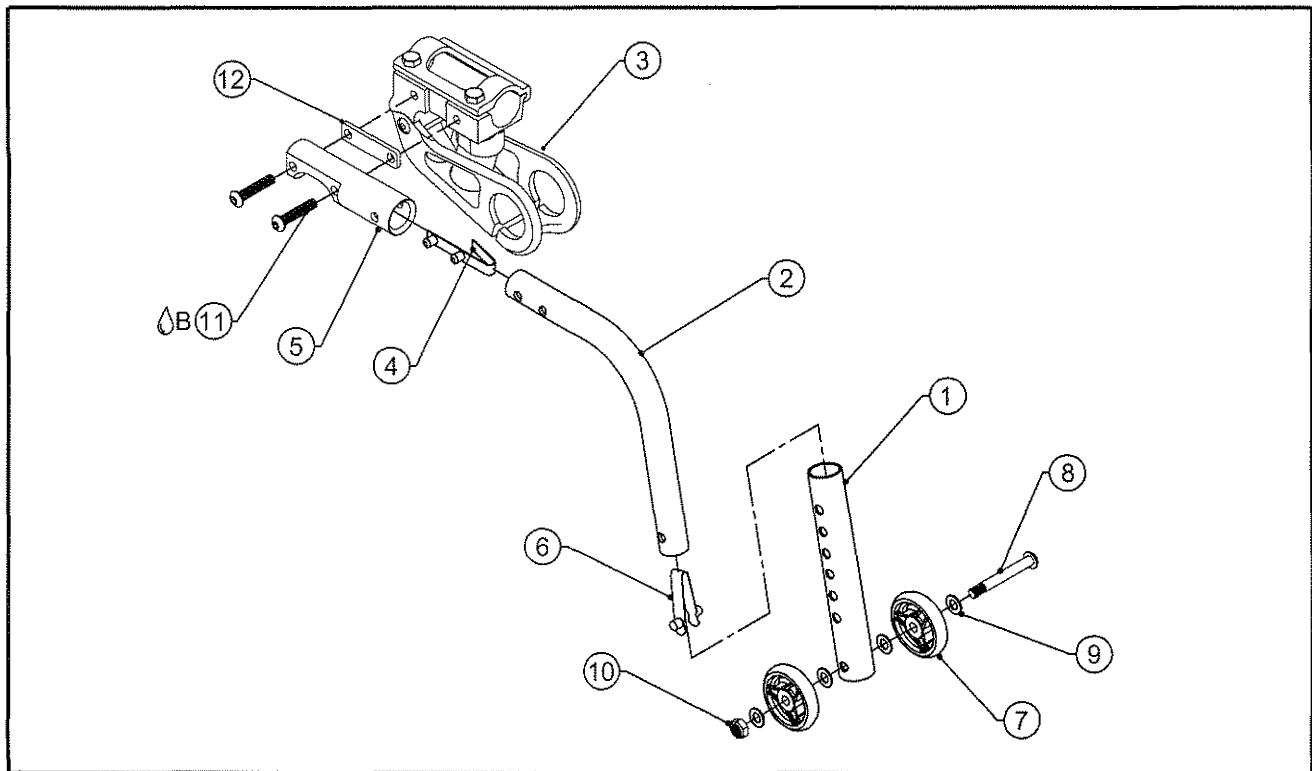

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
2B	1	CP100919	ANTI-TIP EXT TUBE SHORT
2C	1	CP100918	ANTI-TIP EXT TUBE LONG
3A	1	CP103126	FROG LEG® SUSPENSION - USERS UNDER 180 LBS. (PAIR)
3B	1	CP103127	FROG LEG® SUSPENSION - USERS OVER 180 LBS. (PAIR)
4	1	HD100232	SINGLE END DUAL SNAP BUTTON
5	1	CP101045	ANTI-TIP MOUNT
6	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
7	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
8	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
9	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
10	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
11  B	2	HD100342	1/4-28 X 1-1/4" BUTTON HEAD CAP SCREW, BLACK ZINC
12	1	CP103465	ANTI-TIP SPACER FOR FROG REAR SUSPENSION
2A,4,6		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
2B,4,6		AS100570	ANTI-TIP TUBE, SHORT ASSEMBLY
2C,4,6		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,7-10		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,7-10		AS103462	ANTI-TIP EXTENSION ASSY, LONG
1A,2A,3A,4-12		AS104520	EXTRA SHORT ANTI-TIP W/ SHORT EXT - UNDER 180LBS (PAIR)
1A,2A,3B,4-12		AS104521	EXTRA SHORT ANTI-TIP W/ SHORT EXT - OVER 180LBS (PAIR)
1B,2A,3A,4-12		AS104522	EXTRA SHORT ANTI-TIP W/ LONG EXT - UNDER 180LBS (PAIR)
1B,2A,3B,4-12		AS104523	EXTRA SHORT ANTI-TIP W/ LONG EXT - OVER 180LBS (PAIR)
1A,2B,3A,4-12		AS103716	SHORT ANTI-TIP W/ SHORT EXT - UNDER 180LBS (PAIR)
1A,2B,3B,4-12		AS103717	SHORT ANTI-TIP W/ SHORT EXT - OVER 180LBS (PAIR)
1B,2B,3A,4-12		AS103718	SHORT ANTI-TIP W/ LONG EXT - UNDER 180LBS (PAIR)
1B,2B,3B,4-12		AS103719	SHORT ANTI-TIP W/ LONG EXT - OVER 180LBS (PAIR)
1A,2C,3A,4-12		AS103720	LONG ANTI-TIP W/ SHORT EXT - UNDER 180LBS (PAIR)
1A,2C,3B,4-12		AS103721	LONG ANTI-TIP W/ SHORT EXT - OVER 180LBS (PAIR)
1B,2C,3A,4-12		AS103722	LONG ANTI-TIP W/LONG EXT - UNDER 180LBS (PAIR)
1B,2C,3B,4-12		AS103723	LONG ANTI-TIP W/LONG EXT - OVER 180LBS (PAIR)

ANTI-TIP ASSEMBLY

TiLite™ Aero X, 2GX, Aero R, Aero X (SERIES 2), 2GX (SERIES 2)

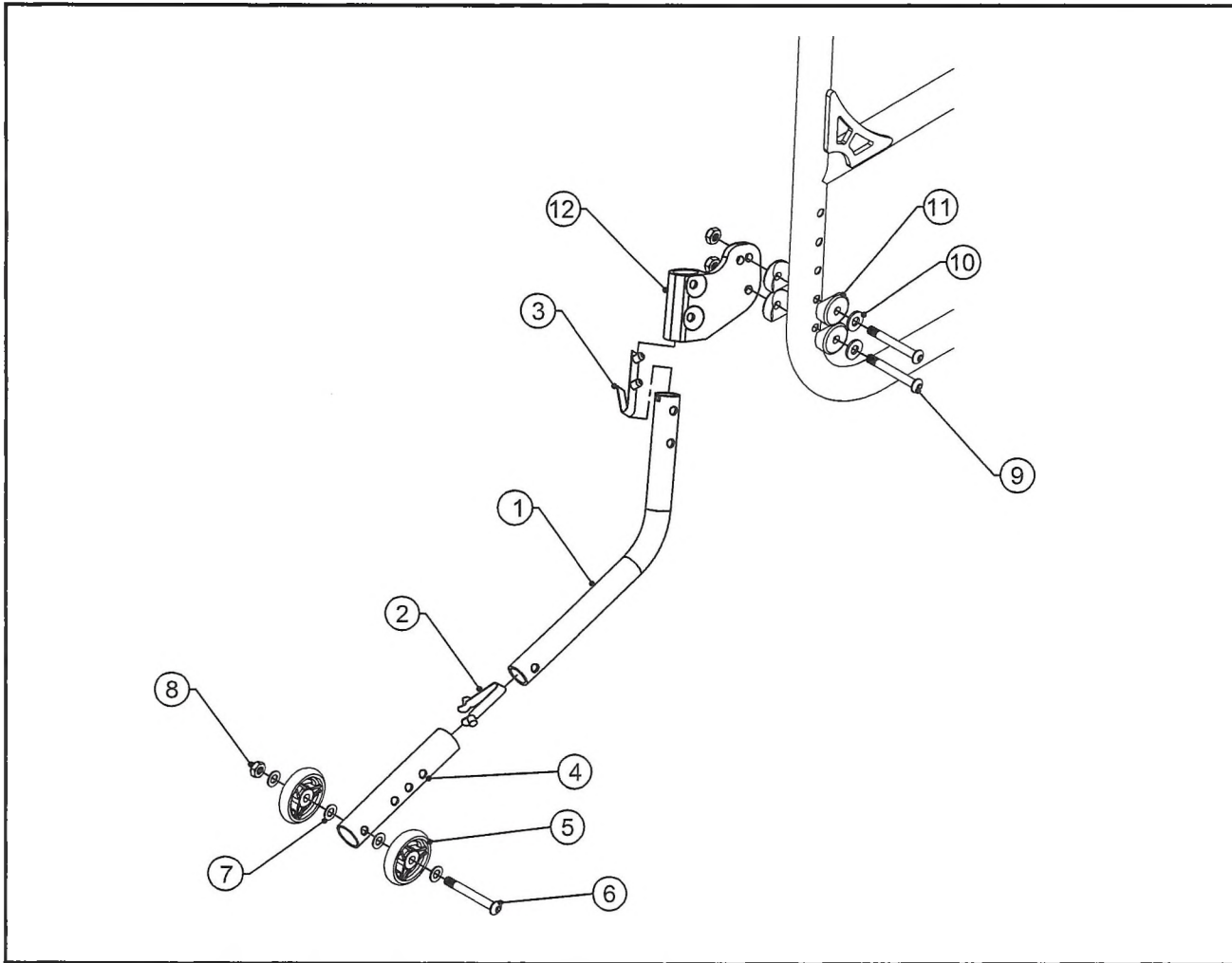

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103136	ANTI TIP BENT TUBE SHORT
1B	1	CP103137	ANTI TIP BENT TUBE
1C	1	CP104586	ANTI-TIP BENT TUBE, LONG
2	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4A	1	CP100550	ANTI-TIP EXTENSION SHORT
4B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
4C	1	CP107544	ANTI-TIP LOWER EXTENSION TUBE EXTRA LONG
5	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
6	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
7	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
8	3	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
9	2	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
10	2	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
11	4	HD100014	COVED WASHER 1" OD X 1/4" ID X 1/8" THK
12A	1	CP107309	FOLDING CHAIR ANTI-TIP RECEPTICAL LEFT
12B	1	CP107308	FOLDING CHAIR ANTI-TIP RECEPTICAL RIGHT
1A-3		AS117459	FOLDING ANTI-TIP BENT TUBE, SHORT ASSEMBLY
1B-3		AS117460	FOLDING ANTI-TIP BENT TUBE, ASSEMBLY
1C-3		AS117461	FOLDING ANTI-TIP BENT TUBE, LONG ASSEMBLY
4A-8		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
4B-8		AS103462	ANTI-TIP EXTENSION ASSY, LONG
4C-8		AS117462	ANTI-TIP EXTENSION ASSY, EXTRA LONG

ASSEMBLIES CONTINUED ON NEXT PAGE

ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ Aero X, 2GX, Aero R, Aero X (SERIES 2), 2GX (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A-3,4A-12A		AS117463	ANTI-TIP BENT SHORT, SHORT EXT ASSEMBLY, LEFT
1A-3,4A-12B		AS117464	ANTI-TIP BENT SHORT, SHORT EXT ASSEMBLY, RIGHT
1A-3,4A-12		AS117465	ANTI-TIP BENT SHORT, SHORT EXT ASSEMBLY (PAIR)
1A-3,4B-12A		AS117466	ANTI-TIP BENT SHORT, LONG EXT ASSEMBLY, LEFT
1A-3,4B-12B		AS117467	ANTI-TIP BENT SHORT, LONG EXT ASSEMBLY, RIGHT
1A-3,4B-12		AS117468	ANTI-TIP BENT SHORT, LONG EXT ASSEMBLY (PAIR)
1A-3,4C-12A		AS117469	ANTI-TIP BENT SHORT, EXTRA LONG EXT ASSEMBLY, LEFT
1A-3,4C-12B		AS117470	ANTI-TIP BENT SHORT, EXTRA LONG EXT ASSEMBLY, RIGHT
1A-3,4C-12		AS117471	ANTI-TIP BENT SHORT, EXTRA LONG EXT ASSEMBLY (PAIR)
1B-3,4A-12A		AS117472	ANTI-TIP BENT TUBE, SHORT EXT ASSEMBLY, LEFT
1B-3,4A-12B		AS117473	ANTI-TIP BENT TUBE, SHORT EXT ASSEMBLY, RIGHT
1B-3,4A-12		AS117474	ANTI-TIP BENT TUBE TUBE, SHORT EXT ASSEMBLY (PAIR)
1B-3,4B-12A		AS117475	ANTI-TIP BENT TUBE, LONG EXT ASSEMBLY, LEFT
1B-3,4B-12B		AS117476	ANTI-TIP BENT TUBE, LONG EXT ASSEMBLY, RIGHT
1B-3,4B-12		AS117477	ANTI-TIP BENT TUBE, LONG EXT ASSEMBLY (PAIR)
1B-3,4C-12A		AS117478	ANTI-TIP BENT TUBE, EXTRA LONG EXT ASSEMBLY, LEFT
1B-3,4C-12B		AS117479	ANTI-TIP BENT TUBE, EXTRA LONG EXT ASSEMBLY, RIGHT
1B-3,4C-12		AS117480	ANTI-TIP BENT TUBE, EXTRA LONG EXT ASSEMBLY (PAIR)
1C-3,4A-12A		AS117481	ANTI-TIP BENT LONG, SHORT EXT ASSEMBLY, LEFT
1C-3,4A-12B		AS117482	ANTI-TIP BENT LONG, SHORT EXT ASSEMBLY, RIGHT
1C-3,4A-12		AS117483	ANTI-TIP BENT LONG, SHORT EXT ASSEMBLY (PAIR)
1C-3,4B-12A		AS117484	ANTI-TIP BENT LONG, LONG EXT ASSEMBLY, LEFT
1C-3,4B-12B		AS117485	ANTI-TIP BENT LONG, LONG EXT ASSEMBLY, RIGHT
1C-3,4B-12		AS117486	ANTI-TIP BENT LONG, LONG EXT ASSEMBLY (PAIR)
1C-3,4C-12A		AS117487	ANTI-TIP BENT LONG, EXTRA LONG EXT ASSEMBLY, LEFT
1C-3,4C-12B		AS117488	ANTI-TIP BENT LONG, EXTRA LONG EXT ASSEMBLY, RIGHT
1C-3,4C-12		AS117489	ANTI-TIP BENT LONG, EXTRA LONG EXT ASSEMBLY (PAIR)



ANTI-TIP ASSEMBLY

TiLite™ X, SX, YG, YGS

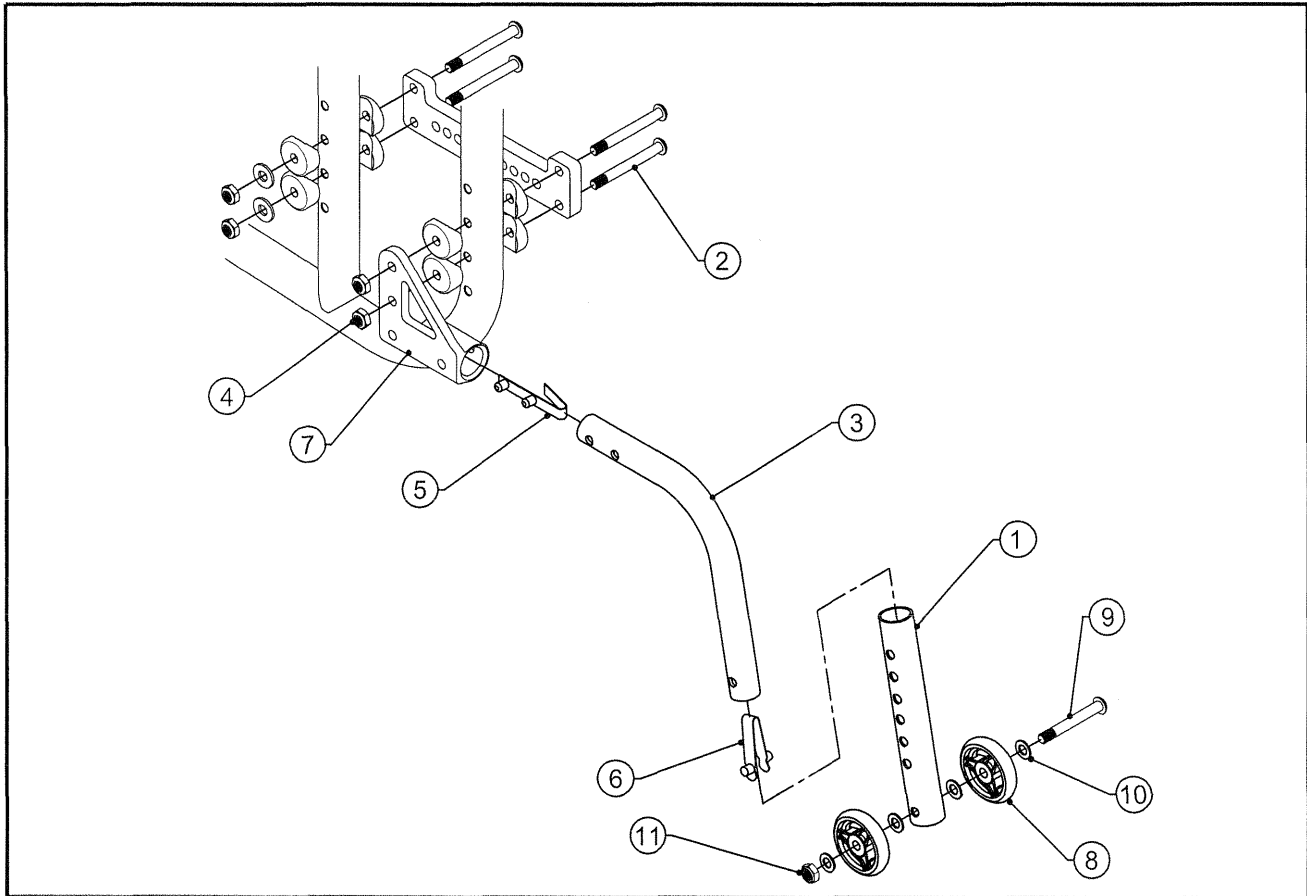

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100550	ANTI-TIP EXTENSION SHORT
1B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
2A	4	CP100241	1/4-28 X 1-5/8" BUTTON HEAD CAP SCREW
2B	4	HD100105	1/4-28 X 2-1/4" BUTTON HEAD CAP SCREW
3A	1	CP103899	ANTI-TIP EXT TUBE EXTRA SHORT
3B	1	CP100919	ANTI-TIP EXT TUBE SHORT
3C	1	CP100918	ANTI-TIP EXT TUBE LONG
4	2	HD100021	1/4-28 X .218 NYLOCK LOCKNUT
5	1	HD100232	SINGLE END DUAL SNAP BUTTON
6	1	HD100155	SINGLE END SNAP BUTTON
7A	1	CP100916	ANTI-TIP MOUNT LEFT
7B	1	CP100917	ANTI-TIP MOUNT RIGHT
8	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
9	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
10	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
11	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
3A,5-6		AS104503	ANTI-TIP TUBE, EXTRA SHORT ASSEMBLY
3B,5-6		AS100570	ANTI-TIP TUBE, SHORT ASSEMBLY
3C,5-6		AS100571	ANTI-TIP TUBE, LONG ASSEMBLY
1A,8-11		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1B,8-11		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE


ANTI-TIP ASSEMBLY (CONTINUED)

TiLite™ X, SX, YG, YGS

ITEM	QTY	PART NO.	DESCRIPTION
1A,2B,3A,4-7A,8-11		AS104504	ANTI-TIP EX SHORT W/SHORT EXT FOR ARMREST, LEFT
1A,2B,3A,4-7B,8-11		AS104505	ANTI-TIP EX SHORT W/SHORT EXT FOR ARMREST, RIGHT
1A,2B,3A,4-11		AS104506	ANTI-TIP EX SHORT W/SHORT EXT FOR ARMREST (PAIR)
1A,2B,3B,4-7A,8-11		AS104456	ANTI-TIP SHORT W/SHORT EXT FOR ARMREST, LEFT
1A,2B,3B,4-7B,8-11		AS104457	ANTI-TIP SHORT W/SHORT EXT FOR ARMREST, RIGHT
1A,2B,3B,4-11		AS100566	ANTI-TIP SHORT W/SHORT EXT FOR ARMREST (PAIR)
1A,2B,3C,4-7A,8-11		AS104458	ANTI-TIP LONG W/SHORT EXT FOR ARMREST, LEFT
1A,2B,3C,4-7B,8-11		AS104459	ANTI-TIP LONG W/SHORT EXT FOR ARMREST, RIGHT
1A,2B,3C,4-11		AS100567	ANTI-TIP LONG W/SHORT EXT FOR ARMREST (PAIR)
1B,2B,3A,4-7A,8-11		AS104507	ANTI-TIP EX SHORT W/LONG EXT FOR ARMREST, LEFT
1B,2B,3A,4-7B,8-11		AS104508	ANTI-TIP EX SHORT W/LONG EXT FOR ARMREST, RIGHT
1B,2B,3A,4-11		AS104509	ANTI-TIP EX SHORT W/LONG EXT FOR ARMREST (PAIR)
1B,2B,3B,4-7A,8-11		AS104460	ANTI-TIP SHORT W/LONG EXT FOR ARMREST, LEFT
1B,2B,3B,4-7B,8-11		AS104461	ANTI-TIP SHORT W/LONG EXT FOR ARMREST, RIGHT
1B,2B,3B,4-11		AS100568	ANTI-TIP SHORT W/LONG EXT FOR ARMREST (PAIR)
1B,2B,3C,4-7A,8-11		AS104462	ANTI-TIP LONG W/LONG EXT FOR ARMREST, LEFT
1B,2B,3C,4-7B,8-11		AS104463	ANTI-TIP LONG W/LONG EXT FOR ARMREST, RIGHT
1B,2B,3C,4-11		AS100569	ANTI-TIP LONG W/LONG EXT FOR ARMREST (PAIR)
1A,2A,3A,4-7A,8-11		AS104510	ANTI-TIP EX SHORT W/SHORT EXT NO ARMREST, LEFT
1A,2A,3A,4-7B,8-11		AS104511	ANTI-TIP EX SHORT W/SHORT EXT NO ARMREST, RIGHT
1A,2A,3A,4-11		AS104512	ANTI-TIP EX SHORT W/SHORT EXT NO ARMREST (PAIR)
1A,2A,3B,4-7A,8-11		AS104464	ANTI-TIP SHORT W/SHORT EXT NO ARMREST, LEFT
1A,2A,3B,4-7B,8-11		AS104465	ANTI-TIP SHORT W/SHORT EXT NO ARMREST, RIGHT
1A,2A,3B,4-11		AS100572	ANTI-TIP SHORT W/SHORT EXT NO ARMREST (PAIR)
1A,2A,3C,4-7A,8-11		AS104466	ANTI-TIP LONG W/SHORT EXT NO ARMREST, LEFT
1A,2A,3C,4-7B,8-11		AS104467	ANTI-TIP LONG W/SHORT EXT NO ARMREST, RIGHT
1A,2A,3C,4-11		AS100573	ANTI-TIP LONG W/SHORT EXT NO ARMREST (PAIR)
1B,2A,3A,4-7A,8-11		AS104513	ANTI-TIP EX SHORT W/LONG EXT NO ARMREST, LEFT
1B,2A,3A,4-7B,8-11		AS104514	ANTI-TIP EX SHORT W/LONG EXT NO ARMREST, RIGHT
1B,2A,3A,4-11		AS104515	ANTI-TIP EX SHORT W/LONG EXT NO ARMREST (PAIR)
1B,2A,3B,4-7A,8-11		AS104468	ANTI-TIP SHORT W/LONG EXT NO ARMREST, LEFT
1B,2A,3B,4-7B,8-11		AS104469	ANTI-TIP SHORT W/LONG EXT NO ARMREST, RIGHT
1B,2A,3B,4-11		AS100574	ANTI-TIP SHORT W/LONG EXT NO ARMREST (PAIR)
1B,2A,3C,4-7A,8-11		AS104470	ANTI-TIP LONG W/LONG EXT NO ARMREST, LEFT
1B,2A,3C,4-7B,8-11		AS104471	ANTI-TIP LONG W/LONG EXT NO ARMREST, RIGHT
1B,2A,3C,4-11		AS100575	ANTI-TIP LONG W/LONG EXT NO ARMREST (PAIR)

ANTI-TIP ASSEMBLY


TiLite™ XC, TX

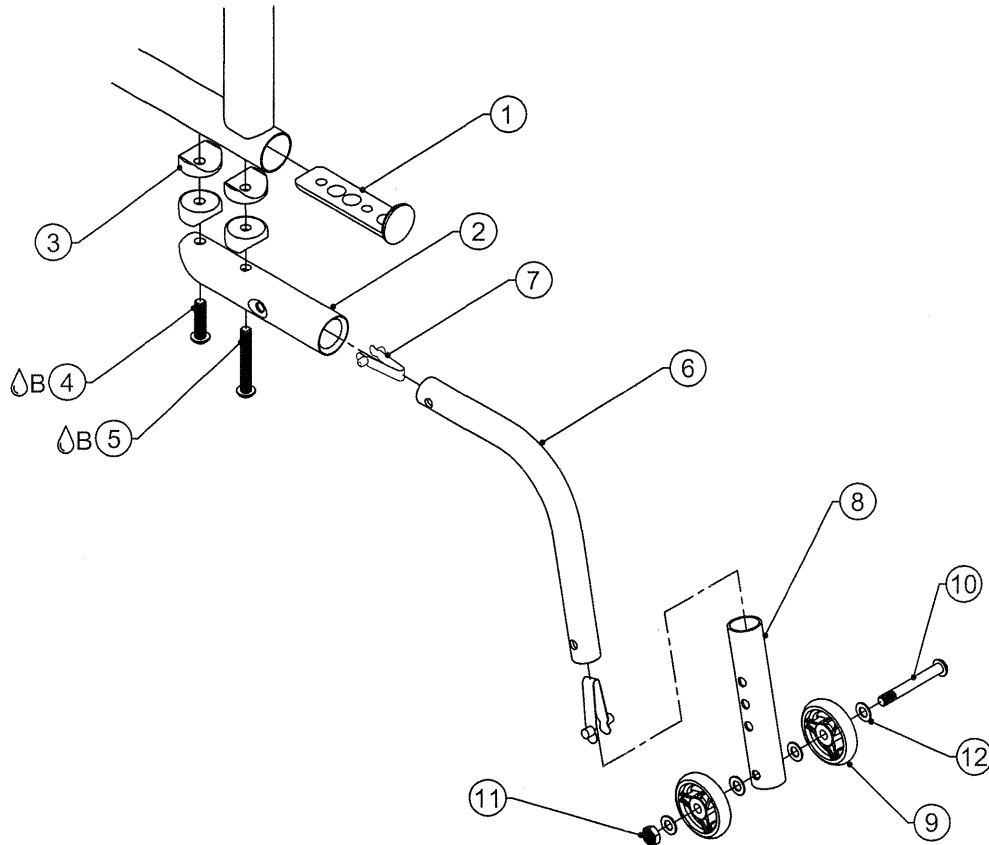

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100775	ANTI-TIP END PLUG
2	1	CP100776	ANTI-TIP RECEPTACLE
3	4	HD100014	COVERED WASHER 1" OD X 1/4" ID X 1/8" THK
4 [Ⓟ] B	1	HD100065	1/4-20 X 1" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
5 [Ⓟ] B	1	HD100125	1/4-20 X 1-3/4" BUTTON HEAD CAP SCREW
6A	1	CP100549	ANTI-TIP TUBE MEDIUM
6B	1	CP100548	ANTI-TIP TUBE SHORT
7	2	CP100251	SNAP BUTTONS, .240 O.D. X .290 TALL
8	1	CP100550	ANTI-TIP EXTENSION SHORT
9	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
10	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
11	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
12	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
1-5		AS100507	ANTI-TIP MOUNT ASSEMBLY
6A,7		AS100456	ANTI-TIP TUBE MEDIUM ASSEMBLY
6B,7		AS100454	ANTI-TIP TUBE SHORT ASSEMBLY
8-12		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
1-5,6A,7-12		AS104477	ANTI-TIP ASSEMBLY MEDIUM
1-5,6A,7-12		AS100502	ANTI-TIP ASSEMBLY MEDIUM (PAIR)
1-5,6B,7-12		AS104478	ANTI-TIP ASSEMBLY SHORT
1-5,6B,7-12		AS100501	ANTI-TIP ASSEMBLY SHORT (PAIR)

ANTI-TIPS ASSY W/ AMPUTEE AXLE PLATE

TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

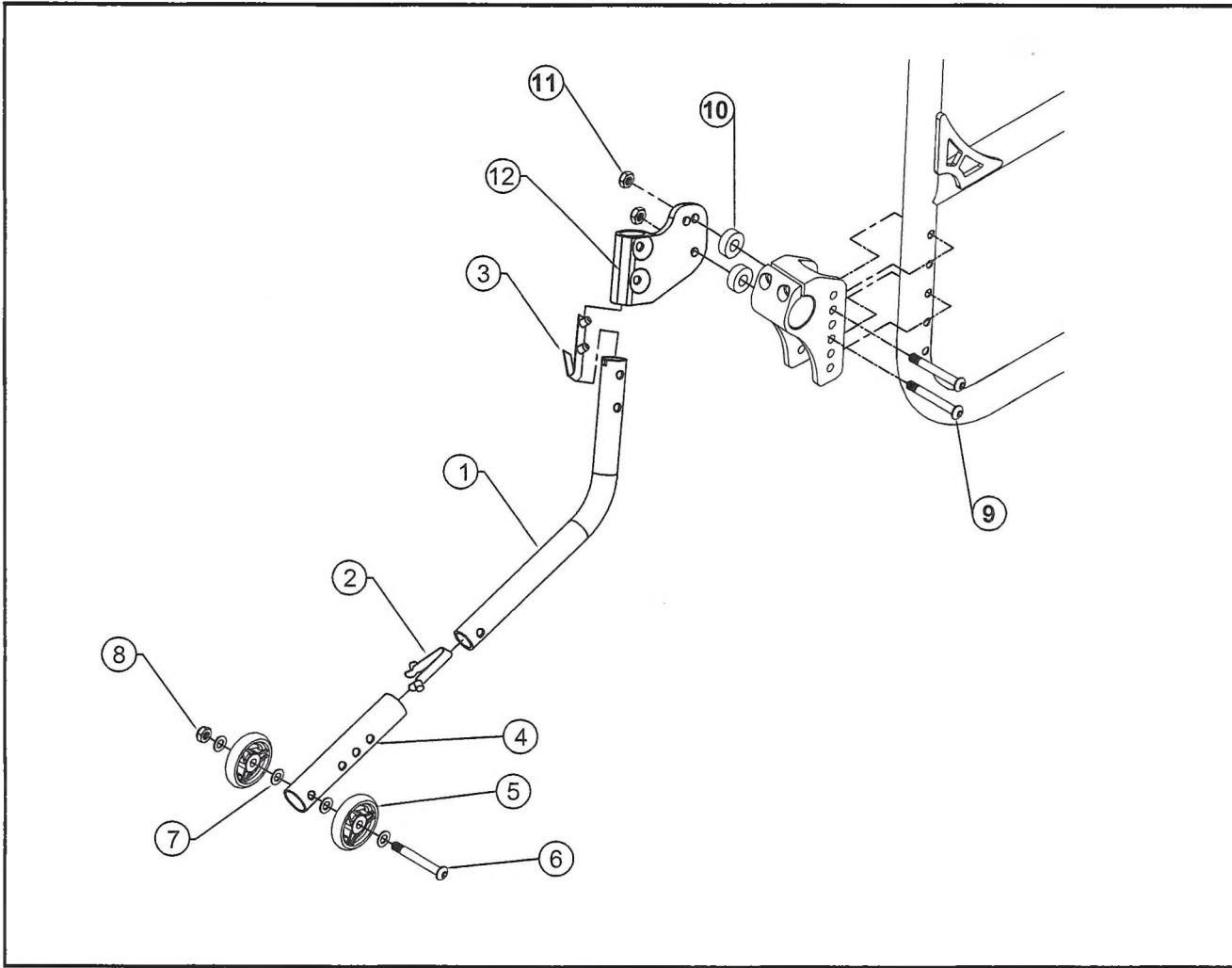

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103136	ANTI TIP BENT TUBE SHORT
1B	1	CP103137	ANTI TIP BENT TUBE
1C	1	CP104586	ANTI-TIP BENT TUBE, LONG
2	1	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
3	1	HD100232	SINGLE END DUAL SNAP BUTTON
4A	1	CP100550	ANTI-TIP EXTENSION SHORT
4B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
4C	1	CP107544	ANTI-TIP LOWER EXTENSION TUBE EXTRA LONG
5	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
6	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
7	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
8	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
9	2	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
10	2	CP102035	NYLON SPACER .25ID X .75OD X .25TH
11	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
12A	1	CP107309	FOLDING CHAIR ANTI-TIP RECEPTICAL LEFT
12B	1	CP107308	FOLDING CHAIR ANTI-TIP RECEPTICAL RIGHT
1A-3		AS117459	FOLDING ANTI-TIP BENT TUBE, SHORT ASSEMBLY
1B-3		AS117460	FOLDING ANTI-TIP BENT TUBE, ASSEMBLY
1C-3		AS117461	FOLDING ANTI-TIP BENT TUBE, LONG ASSEMBLY
4A-8		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
4B-8		AS103462	ANTI-TIP EXTENSION ASSY, LONG
4C-8		AS117462	ANTI-TIP EXTENSION ASSY, EXTRA LONG

ASSEMBLIES CONTINUED ON NEXT PAGE



ANTI-TIP AMPUTEE ASSEMBLY (CONTINUED)


TiLite™ Aero X, 2GX, Aero R, Aero X (SERIES 2), 2GX (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A-3,4A-12A		AS117490	ANTI-TIP AMPUTEE BENT SHORT, SHORT EXT ASSEMBLY, LEFT
1A-3,4A-12B		AS117491	ANTI-TIP AMPUTEE BENT SHORT, SHORT EXT ASSEMBLY, RIGHT
1A-3,4A-12		AS117492	ANTI-TIP AMPUTEE BENT SHORT, SHORT EXT ASSEMBLY, PAIR
1A-3,4B-12A		AS117493	ANTI-TIP AMPUTEE BENT SHORT, LONG EXT ASSEMBLY, LEFT
1A-3,4B-12B		AS117494	ANTI-TIP AMPUTEE BENT SHORT, LONG EXT ASSEMBLY, RIGHT
1A-3,4B-12		AS117495	ANTI-TIP AMPUTEE BENT SHORT, LONG EXT ASSEMBLY, PAIR
1A-3,4C-12A		AS117496	ANTI-TIP AMPUTEE BENT SHORT, EXTRA LONG EXT ASSEMBLY, LEFT
1A-3,4C-12B		AS117497	ANTI-TIP AMPUTEE BENT SHORT, EXTRA LONG EXT ASSEMBLY, RIGHT
1A-3,4C-12		AS117498	ANTI-TIP AMPUTEE BENT SHORT, EXTRA LONG EXT ASSEMBLY, PAIR
1B-3,4A-12A		AS117499	ANTI-TIP AMPUTEE BENT TUBE, SHORT EXT ASSEMBLY, LEFT
1B-3,4A-12B		AS117500	ANTI-TIP AMPUTEE BENT TUBE, SHORT EXT ASSEMBLY, RIGHT
1B-3,4A-12		AS117501	ANTI-TIP AMPUTEE BENT TUBE, SHORT EXT ASSEMBLY, PAIR
1B-3,4B-12A		AS117502	ANTI-TIP AMPUTEE BENT TUBE, LONG EXT ASSEMBLY, LEFT
1B-3,4B-12B		AS117503	ANTI-TIP AMPUTEE BENT TUBE, LONG EXT ASSEMBLY, RIGHT
1B-3,4B-12		AS117504	ANTI-TIP AMPUTEE BENT TUBE, LONG EXT ASSEMBLY, PAIR
1B-3,4C-12A		AS117505	ANTI-TIP AMPUTEE BENT TUBE, EXTRA LONG EXT ASSEMBLY, LEFT
1B-3,4C-12B		AS117506	ANTI-TIP AMPUTEE BENT TUBE, EXTRA LONG EXT ASSEMBLY, RIGHT
1B-3,4C-12		AS117507	ANTI-TIP AMPUTEE BENT TUBE, EXTRA LONG EXT ASSEMBLY, PAIR
1C-3,4A-12A		AS117508	ANTI-TIP AMPUTEE BENT LONG, SHORT EXT ASSEMBLY, LEFT
1C-3,4A-12B		AS117509	ANTI-TIP AMPUTEE BENT LONG, SHORT EXT ASSEMBLY, RIGHT
1C-3,4A-12		AS117510	ANTI-TIP AMPUTEE BENT LONG, SHORT EXT ASSEMBLY, PAIR
1C-3,4B-12A		AS117511	ANTI-TIP AMPUTEE BENT LONG, LONG EXT ASSEMBLY, LEFT
1C-3,4B-12B		AS117512	ANTI-TIP AMPUTEE BENT LONG, LONG EXT ASSEMBLY, RIGHT
1C-3,4B-12		AS117513	ANTI-TIP AMPUTEE BENT LONG, LONG EXT ASSEMBLY, PAIR
1C-3,4C-12A		AS117514	ANTI-TIP AMPUTEE BENT LONG, EXTRA LONG EXT ASSEMBLY, LEFT
1C-3,4C-12B		AS117515	ANTI-TIP AMPUTEE BENT LONG, EXTRA LONG EXT ASSEMBLY, RIGHT
1C-3,4C-12		AS117516	ANTI-TIP AMPUTEE BENT LONG, EXTRA LONG EXT ASSEMBLY, PAIR



USER FRIENDLY REAR FLIP-UP ANTI-TIP


TiLite™ TR (SERIES 2), YR (SERIES 2)

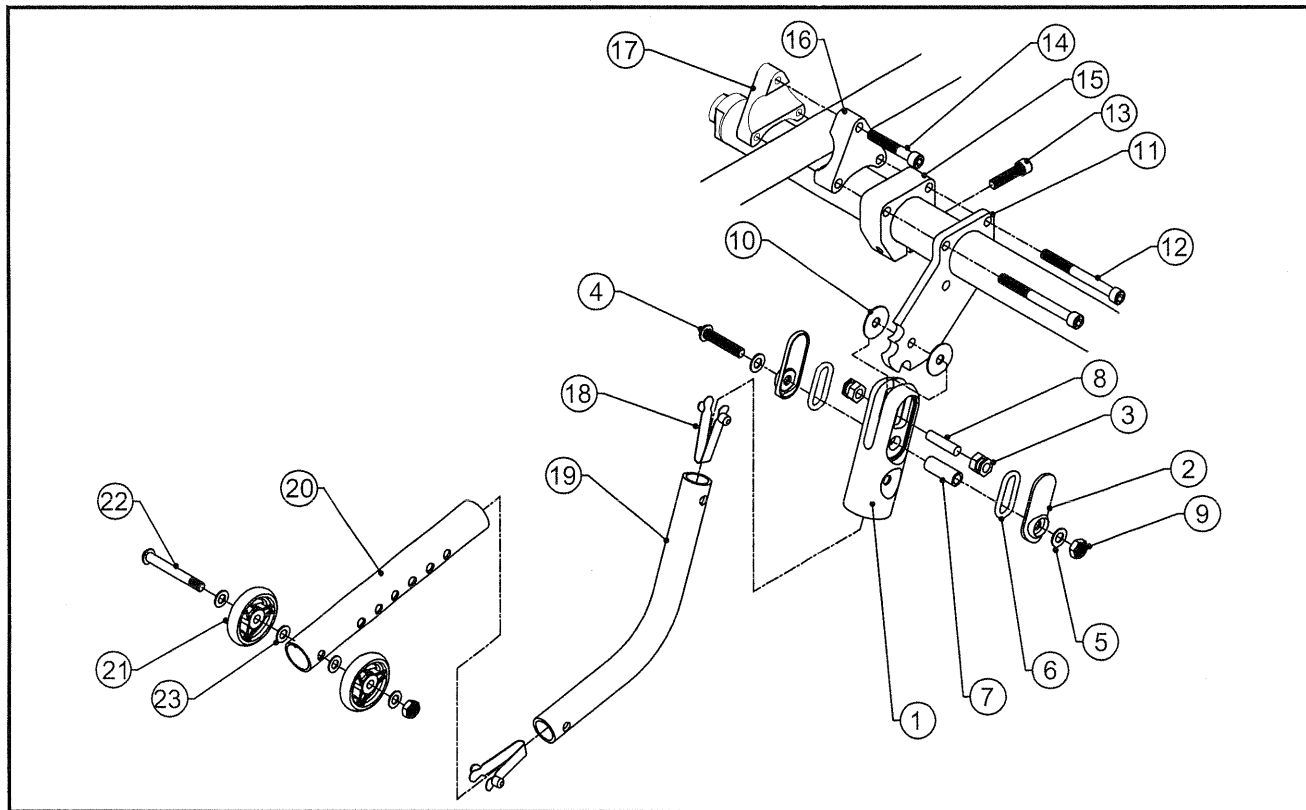

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP103135	ANTI-TIP PLATE
12	2	HD100227	1/4-20 X 2-1/2 LONG SOCKET HEAD CAP SCREW
13	1	HD100051	1/4-20 X 1" SOCKET HEAD CAP SCREW, CLEAR ZINC PLATED
14	1	HD100243	1/4-20 X 1-1/2" SOCKET HEAD CAP SCREW
15	1	CP101063	ADJUSTABLE HEIGHT CAMBER TUBE MOUNT
16	1	CP101060	CAMBER MOUNT NON-THREADED
17	1	CP101061	CAMBER MOUNT THREADED
18	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
19A	1	CP103136	ANTI-TIP BENT TUBE SHORT
19B	1	CP103137	ANTI-TIP BENT TUBE
20A	1	CP100550	ANTI-TIP EXTENSION SHORT
20B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
21	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
22	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
23	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
1-17		AS103085	FLIP-UP ANTI-TIPPER SUB-ASSEMBLY
18-19A		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
18-19B		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
20A-24		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
20B-24		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-19A,20A		AS103137	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT
1-19A,20A		AS103141	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT PAIR
1-19A,20B		AS103138	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG
1-19A,20B		AS103142	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG PAIR
1-19B,20A		AS103139	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT
1-19B,20A		AS103143	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT PAIR
1-19B,20B		AS103140	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG
1-19B,20B		AS103144	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG PAIR

USER FRIENDLY REAR FLIP-UP ANTI-TIP


TiLite™ ZR

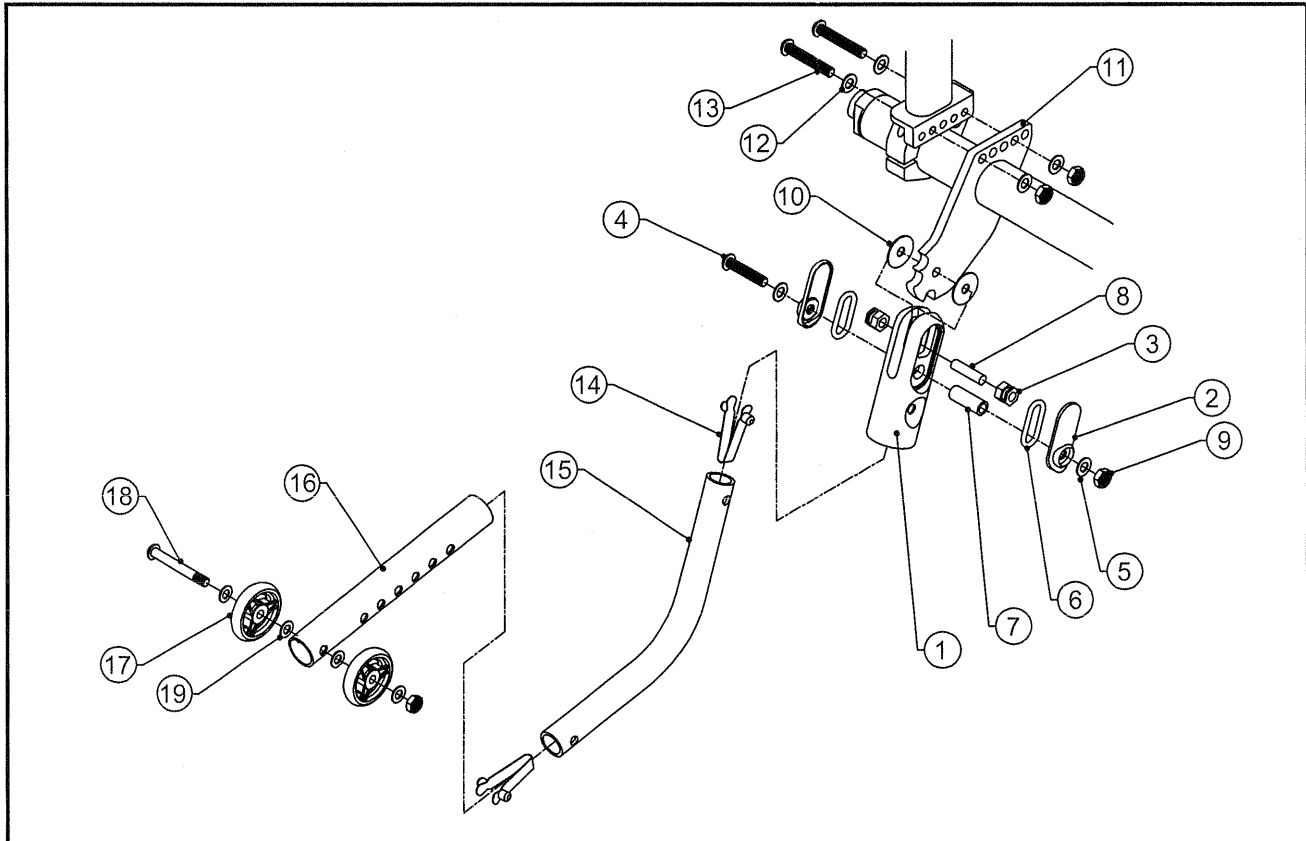

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	4	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP103133	ANTI-TIP PLATE
12	4	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
13	2	HD100440	1/4-28 X 5/8 BUTTON HEAD CAP SCREW
14	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
15A	1	CP103136	ANTI-TIP BENT TUBE SHORT
15B	1	CP103137	ANTI-TIP BENT TUBE
16A	1	CP100550	ANTI-TIP EXTENSION SHORT
16B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
17	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
18	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
19	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
1-13		AS103376	FLIP-UP ANTI-TIP SUB-ASSEMBLY
14-15A		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
14-15B		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
16A-20		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
16B-20		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-15A, 16A-19		AS103378	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT
1-15A, 16A-19		AS103379	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT PAIR
1-15A, 16B-19		AS103380	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG
1-15A, 16B-19		AS103381	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG PAIR
1-15B, 16A-19		AS103382	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT
1-15B, 16A-19		AS103383	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT PAIR
1-15B, 16B-19		AS103384	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG
1-15B, 16B-19		AS103385	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG PAIR

USER FRIENDLY ANTI-TIP ASSEMBLY

TiLite™ ZR (SERIES 2)

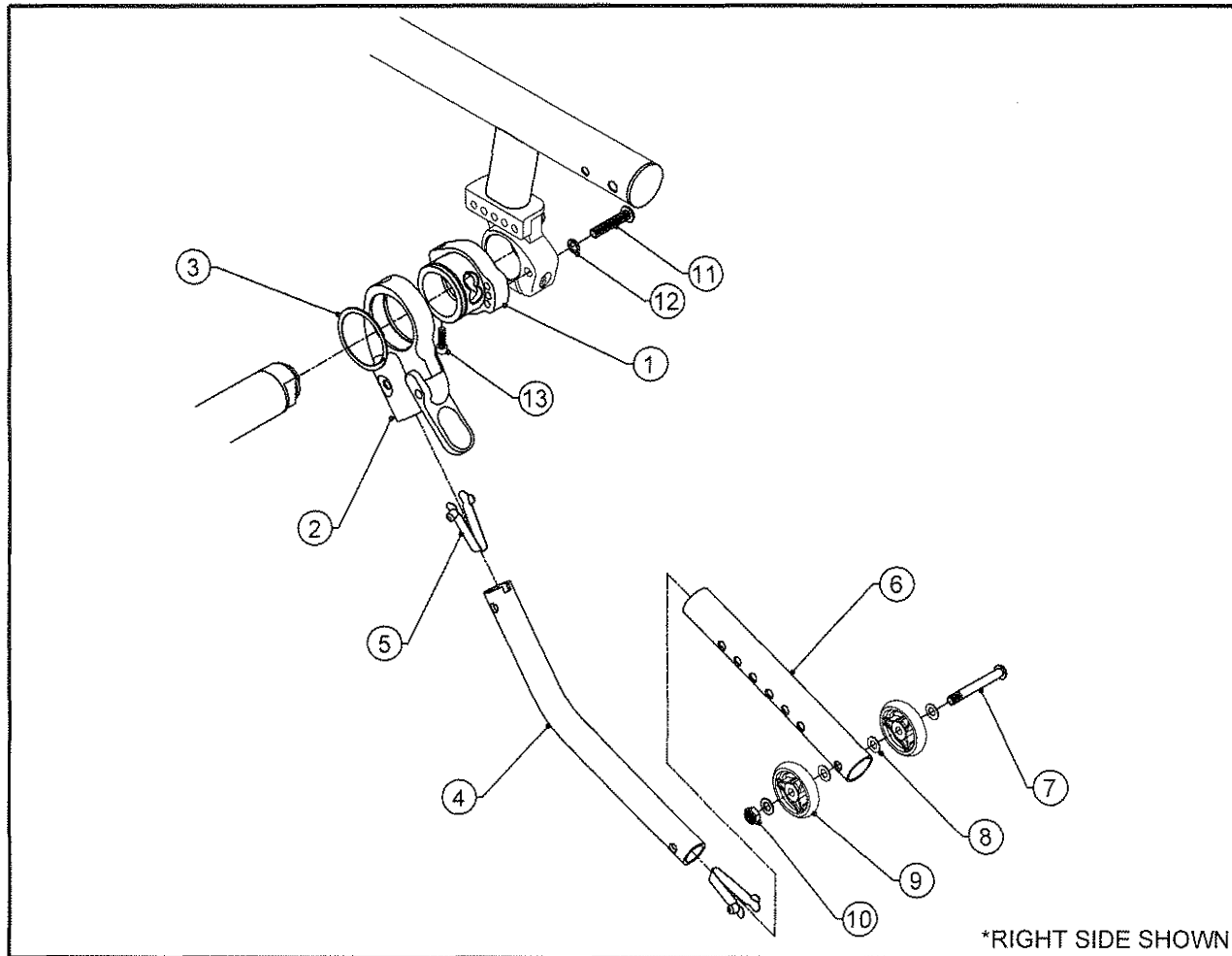

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP104582	ANTI-TIP MOUNT, RIGHT
1B	1	CP104583	ANTI-TIP MOUNT, LEFT
2A	1	AS104972	ANTI-TIP PIVOT SUB-ASSY, RIGHT
2B	1	AS104973	ANTI-TIP PIVOT SUB-ASSY, LEFT
3	1	HD100613	Ø1-5/8" SPIRAL EXTERNAL RETAINING RING
4A	1	CP103136	ANTI-TIP BENT TUBE, SHORT
4B	1	CP103137	ANTI-TIP BENT TUBE
4C	1	CP104586	ANTI-TIP BENT TUBE, LONG
5	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
10	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
11	1	HD100633	1/4-20 X 1-3/8 BUTTON HEAD CAP SCREW, BLACK ZINC
12	1	HD100618	1/4" LOCK WASHER, BLACK ZINC
13	1	HD100341	1/4-20 X 3/4" SHCS, BLACK ZINC
1A,2A-3,13		AS117078	UFAT SUB ASSEMBLY RIGHT
1B,2B-3,13		AS117079	UFAT SUB ASSEMBLY LEFT
4A,5		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
4B,5		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
4C,5		AS104935	ANTI-TIP BENT TUBE LONG ASSEMBLY
6-10		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE

USER FRIENDLY ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ ZR (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3,4A,5-13		AS104936	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4A,5-13		AS104937	SHORT ANTI-TIP W/ LONG EXT, LEFT
1-3,4A,5-13		AS104938	SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2A,3,4B,5-13		AS104939	MEDIUM ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4B,5-13		AS104940	MEDIUM ANTI-TIP W/ LONG EXT, LEFT
1-3,4B,5-13		AS104941	MEDIUM ANTI-TIP W/ LONG EXT (PAIR)
1A,2A,3,4C,5-13		AS104942	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4C,5-13		AS104943	LONG ANTI-TIP W/ LONG EXT, LEFT
1-3,4C,5-13		AS104944	LONG ANTI-TIP W/ LONG EXT (PAIR)



USER FRIENDLY REAR FLIP-UP ANTI-TIP


TiLite™ ZRA, AERO Z

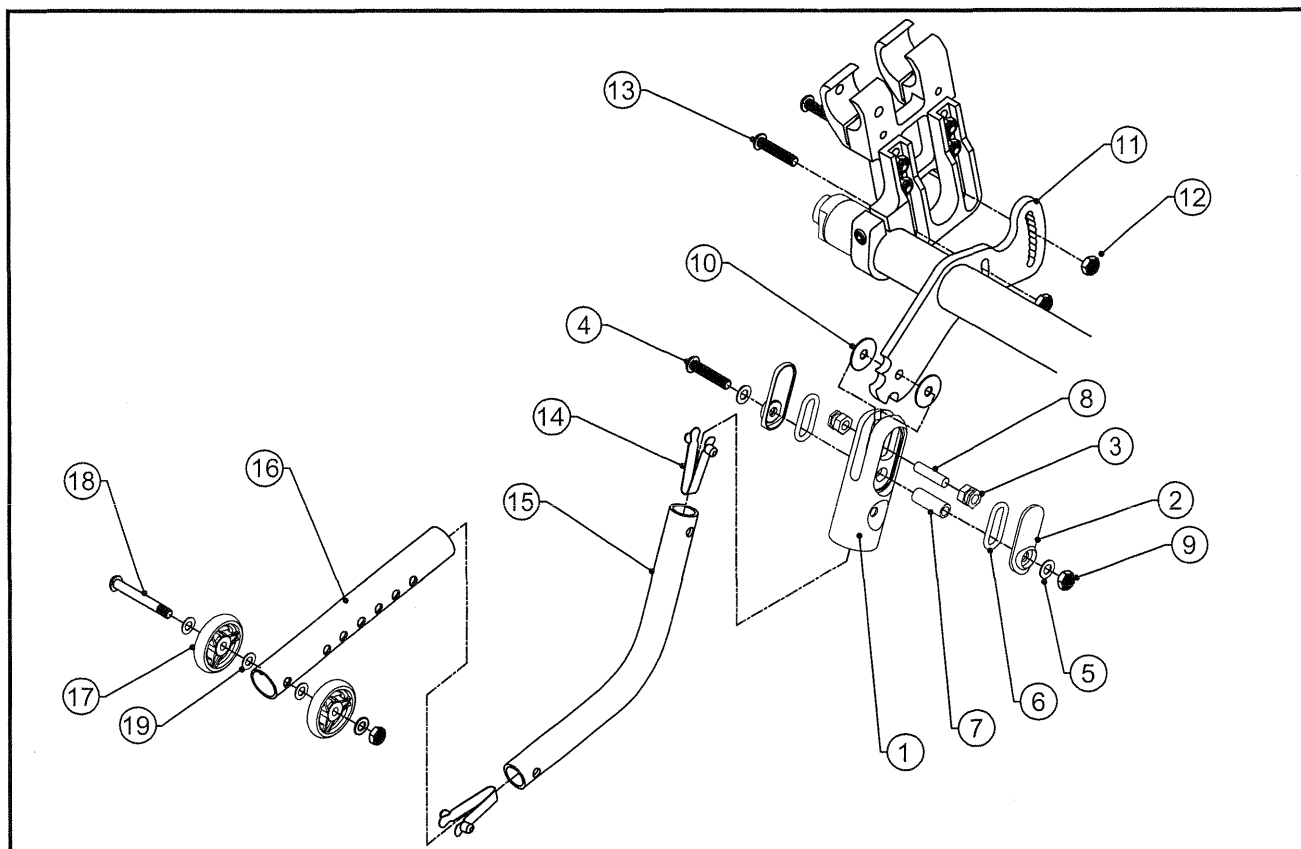

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP103134	ANTI-TIP PLATE
12	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
13	2	HD100438	1/4-28 X 1-1/4" BUTTON HEAD CAP SCREW, BLACK
14	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
15A	1	CP103136	ANTI-TIP BENT TUBE SHORT ASSEMBLY
15B	1	CP103137	ANTI-TIP BENT TUBE ASSEMBLY
16A	1	CP100550	ANTI-TIP EXTENSION SHORT
16B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
17	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
18	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
19	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
1-13		AS103377	FLIP-UP ANTI-TIP SUB-ASSEMBLY
14-15A		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
14-15B		AS103375	ANTI-TIP BENT TUBE LONG ASSEMBLY
16A-19		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
16B-19		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-15A,16A-19		AS103386	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT
1-15A,16A-19		AS103387	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT PAIR
1-15A,16B-19		AS103388	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG
1-15A,16B-19		AS103389	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG PAIR
1-15B,16A-19		AS103390	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT
1-15B,16A-19		AS103391	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT PAIR
1-15B,16B-19		AS103392	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG
1-15B,16B-19		AS103393	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG PAIR

USER FRIENDLY ANTI-TIP ASSEMBLY

TiLite™ ZRA (SERIES 2), AERO Z (SERIES 2), TWIST, AERO T, TRA (SERIES 2)

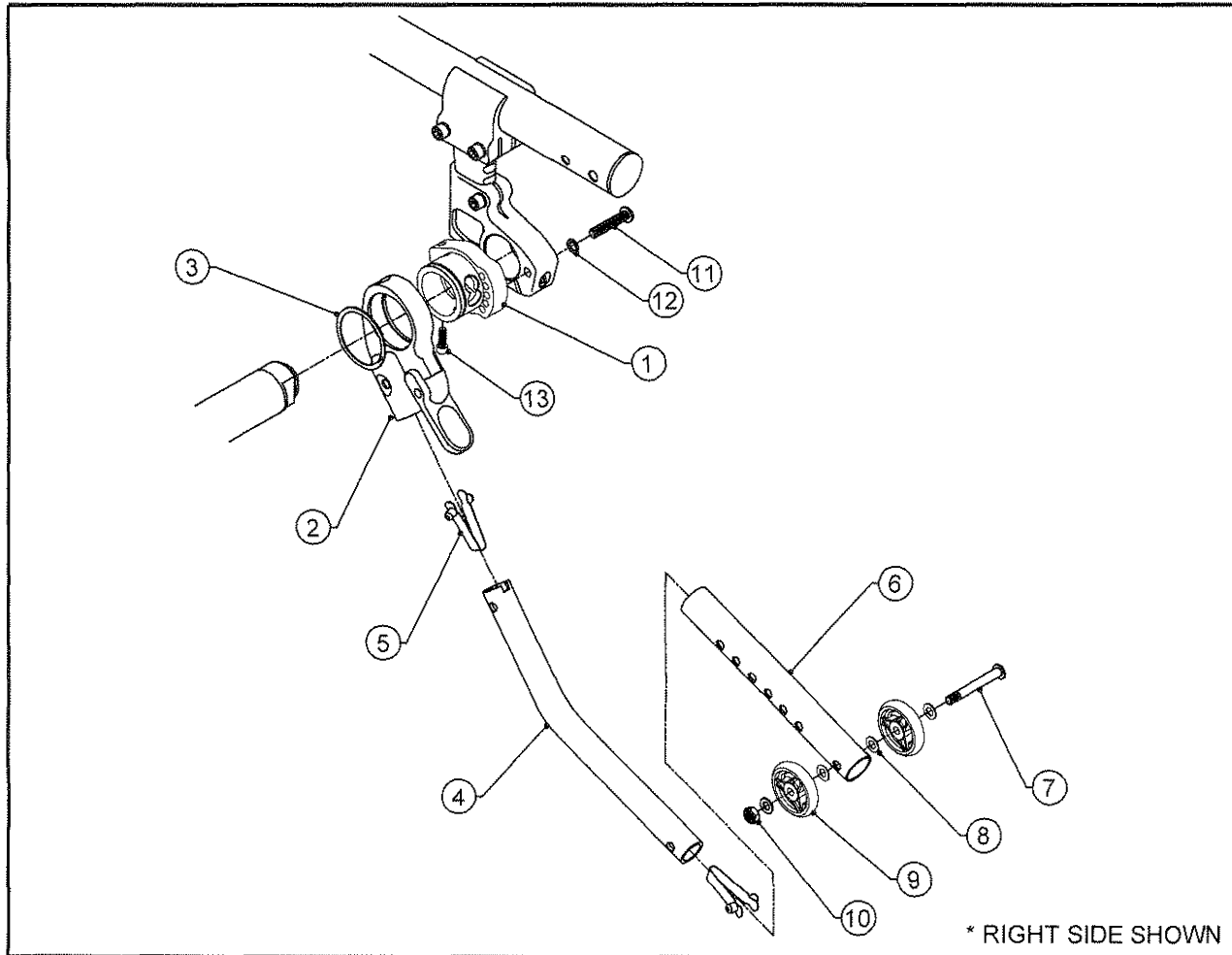

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP104582	ANTI-TIP MOUNT, RIGHT
1B	1	CP104583	ANTI-TIP MOUNT, LEFT
2A	1	AS104972	ANTI-TIP PIVOT SUB-ASSY, RIGHT
2B	1	AS104973	ANTI-TIP PIVOT SUB-ASSY, LEFT
3	1	HD100613	Ø1-5/8" SPIRAL EXTERNAL RETAINING RING
4A	1	CP103136	ANTI-TIP BENT TUBE, SHORT
4B	1	CP103137	ANTI-TIP BENT TUBE
4C	1	CP104586	ANTI-TIP BENT TUBE, LONG
5	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
10	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
11	1	HD100633	1/4-20 X 1-3/8 BUTTON HEAD CAP SCREW, BLACK ZINC
12	1	HD100618	1/4" LOCK WASHER, BLACK ZINC
13	1	HD100341	1/4-20 X 3/4" SHCS, BLACK ZINC
1A,2A-3,13		AS117078	UFAT SUB ASSEMBLY RIGHT
1B,2B-3,13		AS117079	UFAT SUB ASSEMBLY LEFT
4A,5		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
4B,5		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
4C,5		AS104935	ANTI-TIP BENT TUBE LONG ASSEMBLY
6-10		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE

USER FRIENDLY ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ ZRA (SERIES 2), AERO Z (SERIES 2), TWIST, AERO T, TRA (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3,4A,5-13		AS104936	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4A,5-13		AS104937	SHORT ANTI-TIP W/ LONG EXT, LEFT
1-3,4A,5-13		AS104938	SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2A,3,4B,5-13		AS104939	MEDIUM ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4B,5-13		AS104940	MEDIUM ANTI-TIP W/ LONG EXT, LEFT
1-3,4B,5-13		AS104941	MEDIUM ANTI-TIP W/ LONG EXT (PAIR)
1A,2A,3,4C,5-13		AS104942	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4C,5-13		AS104943	LONG ANTI-TIP W/ LONG EXT, LEFT
1-3,4C,5-13		AS104944	LONG ANTI-TIP W/ LONG EXT (PAIR)



USER FRIENDLY ANTI-TIP ASSEMBLY

TiLite™ TR (SERIES 3)

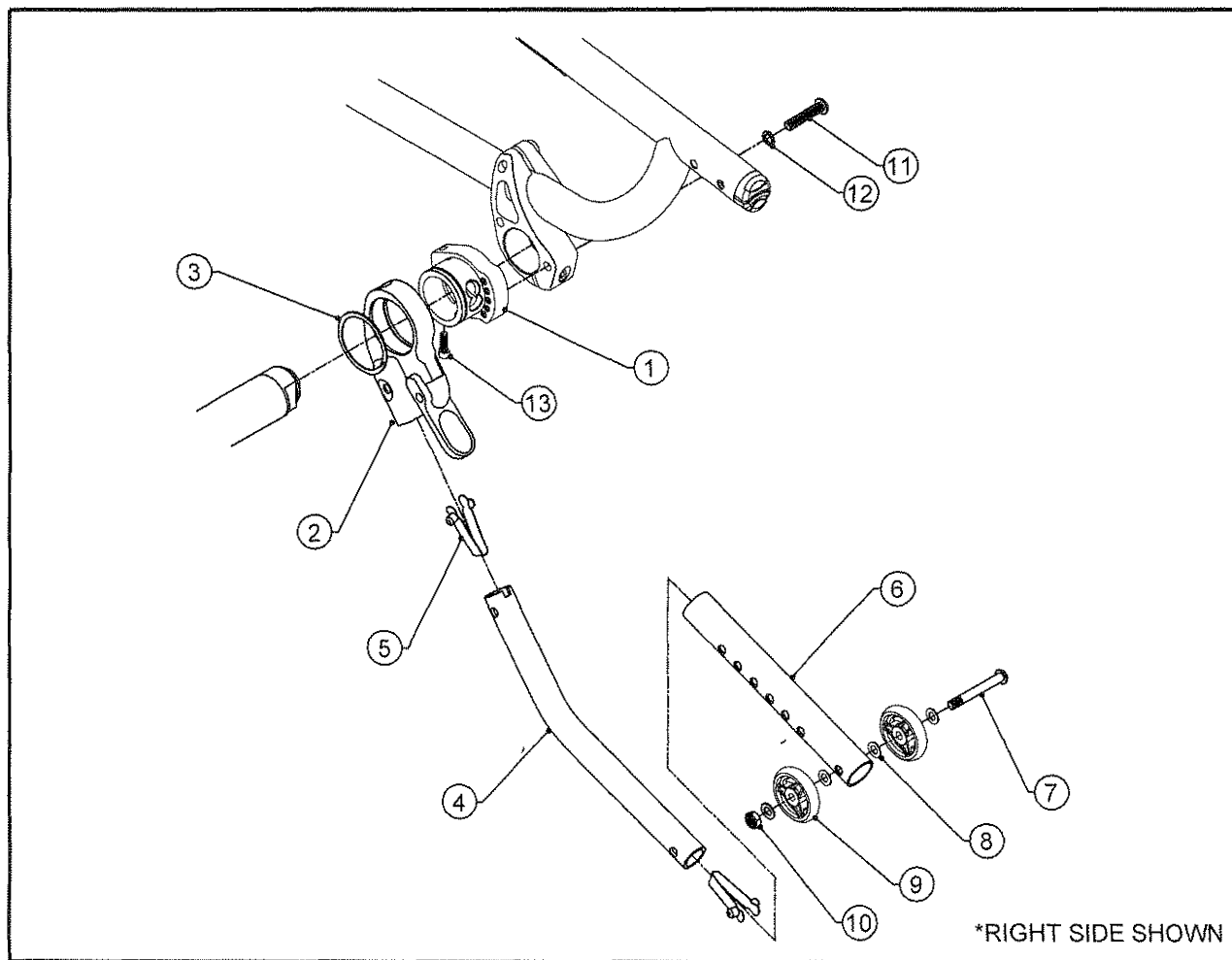

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP104582	ANTI-TIP MOUNT, RIGHT
1B	1	CP104583	ANTI-TIP MOUNT, LEFT
2A	1	AS104972	ANTI-TIP PIVOT SUB-ASSY, RIGHT
2B	1	AS104973	ANTI-TIP PIVOT SUB-ASSY, LEFT
3	1	HD100613	Ø1-5/8" SPIRAL EXTERNAL RETAINING RING
4A	1	CP103136	ANTI-TIP BENT TUBE, SHORT
4B	1	CP103137	ANTI-TIP BENT TUBE
4C	1	CP104586	ANTI-TIP BENT TUBE, LONG
5	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
6	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
7	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
9	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
10	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
11	1	HD100633	1/4-20 X 1-3/8 BUTTON HEAD CAP SCREW, BLACK ZINC
12	1	HD100618	1/4" LOCK WASHER, BLACK ZINC
13	1	HD100341	1/4-20 X 3/4" SHCS, BLACK ZINC
1A,2A-3,13		AS117078	UFAT SUB ASSEMBLY RIGHT
1B,2B-3,13		AS117079	UFAT SUB ASSEMBLY LEFT
4A,5		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
4B,5		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
4C,5		AS104935	ANTI-TIP BENT TUBE LONG ASSEMBLY
6-10		AS103462	ANTI-TIP EXTENSION ASSY, LONG

ASSEMBLIES CONTINUED ON NEXT PAGE

USER FRIENDLY ANTI-TIP ASSEMBLY (CONTINUED)


TiLite™ TR (SERIES 3)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3,4A,5-13		AS104936	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4A,5-13		AS104937	SHORT ANTI-TIP W/ LONG EXT, LEFT
1-3,4A,5-13		AS104938	SHORT ANTI-TIP W/ LONG EXT (PAIR)
1A,2A,3,4B,5-13		AS104939	MEDIUM ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4B,5-13		AS104940	MEDIUM ANTI-TIP W/ LONG EXT, LEFT
1-3,4B,5-13		AS104941	MEDIUM ANTI-TIP W/ LONG EXT (PAIR)
1A,2A,3,4C,5-13		AS104942	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4C,5-13		AS104943	LONG ANTI-TIP W/ LONG EXT, LEFT
1-3,4C,5-13		AS104944	LONG ANTI-TIP W/ LONG EXT (PAIR)



USER FRIENDLY REAR FLIP-UP ANTI-TIP

TiLite™ Aero X, 2GX, Aero R, Aero X (SERIES 2), 2GX (SERIES 2)

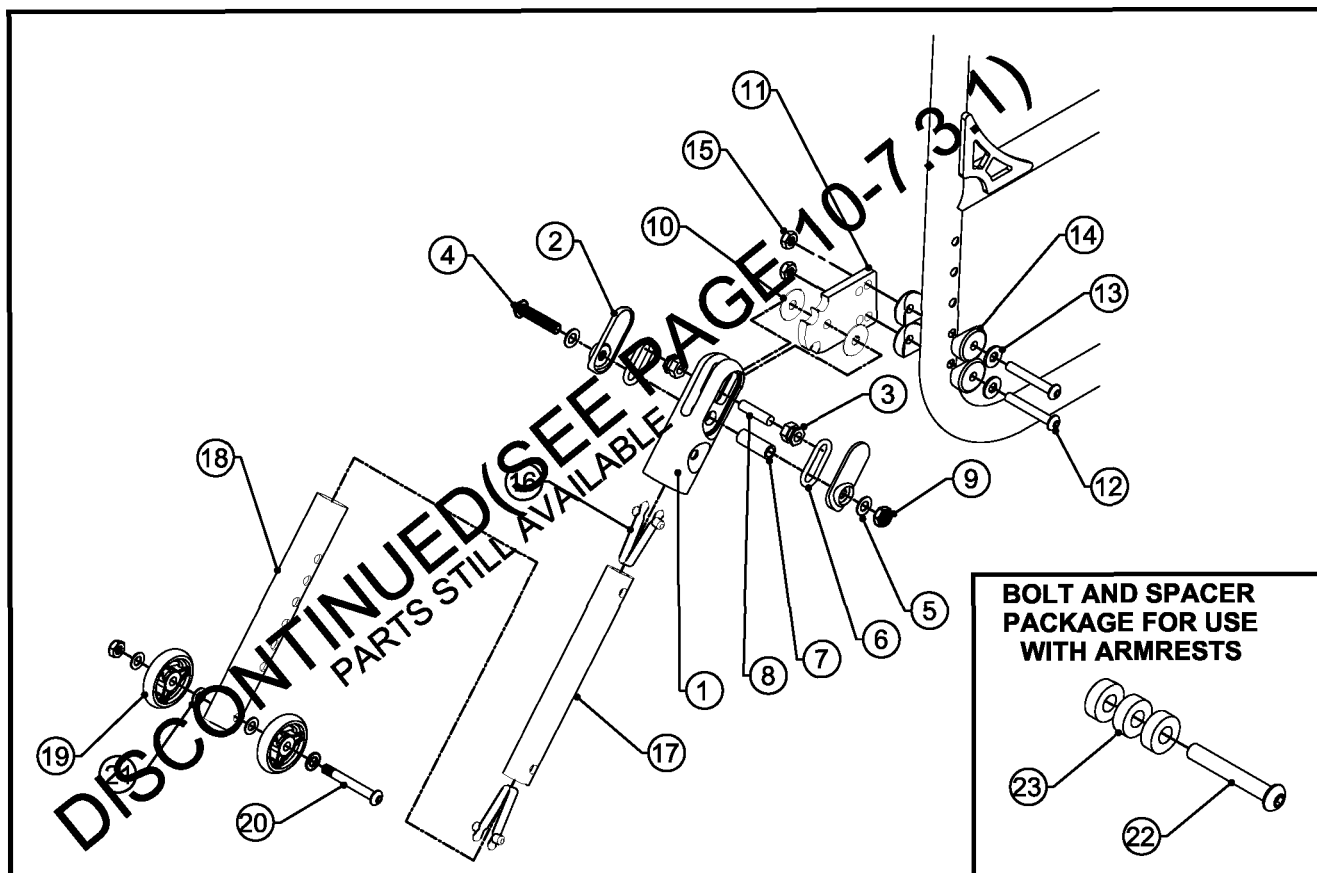


ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP102594	ANTI-TIP PLATE
12	2	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
13	2	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
14	4	HD100014	COVED WASHER 1" OD X 1/4" ID X 1/8" THK
15	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
16	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
17A	1	CP102891	ANTI-TIP STRAIGHT TUBE SHORT
17B	1	CP102892	ANTI-TIP STRAIGHT TUBE LONG
18A	1	CP100550	ANTI-TIP EXTENSION SHORT
18B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
19	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
20	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
21	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
22	2	HD100491	1/4-28 X 2-3/4" BUTTON HEAD CAP SCREW, BLACK ZINC
23	6	CP102035	NYLON SPACER .25ID X .75OD X .25TH
1-11		AS103087	FLIP-UP ANTI-TIP SUB-ASSEMBLY
16-17A		AS103088	ANTI-TIP STRAIGHT TUBE SHORT ASSEMBLY
16-17A		AS103089	ANTI-TIP STRAIGHT TUBE LONG ASSEMBLY
9,18A-21		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
9,18B-21		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-17A,18A-21		AS103595	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT
1-17A,18A-21		AS103599	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT PAIR
1-17A,18B-21		AS103596	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG
1-17A,18B-21		AS103600	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG PAIR
1-17B,18A-21		AS103597	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT
1-17B,18A-21		AS103601	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT PAIR
1-17B,18B-21		AS103598	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG
1-17B,18B-21		AS103602	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG PAIR
22-23		AS103679	FLIP-UP ANTI-TIP ASSEMBLY SPACER PACK



NEW USER FRIENDLY REAR FLIP-UP ANTI-TIP

TiLite™ Aero X, 2GX, Aero R, Aero X (SERIES 2), 2GX (SERIES 2)

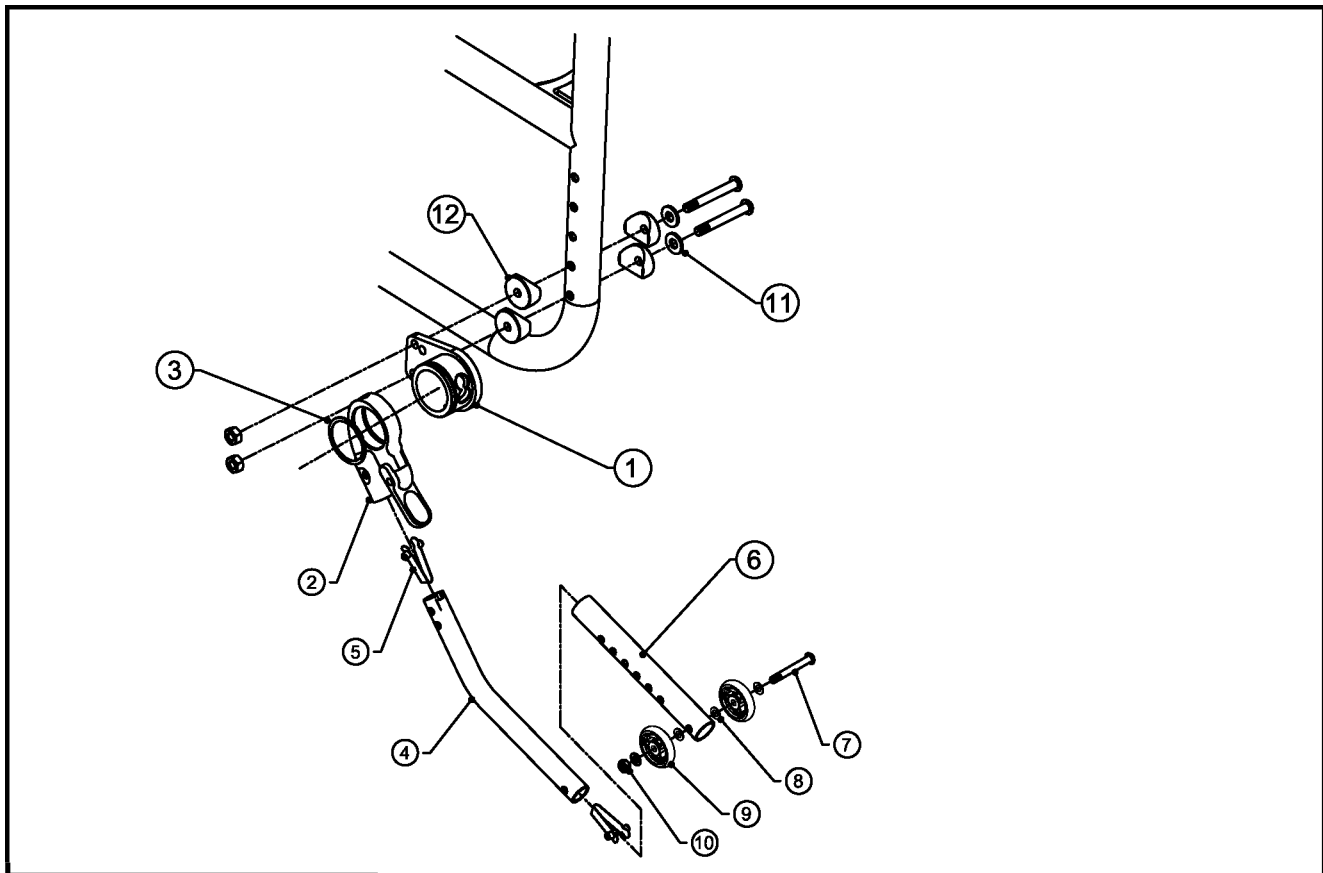

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3		AS117517	FOLDING UFAT SUB ASSEMBLY RIGHT
1B,2B,3		AS117518	FOLDING UFAT SUB ASSEMBLY LEFT
4A	1	CP103136	ANTI TIP BENT TUBE SHORT
4B	1	CP103137	ANTI TIP BENT TUBE
4C	1	CP104586	ANTI-TIP BENT TUBE, LONG
5	2	CP100251	SNAP BUTTON, .240 OD X .290 TALL
6A	1	CP100550	ANTI-TIPPER EXTENSION SHORT
6B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
7	3	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 OD X .265 ID X .031 THICK
9	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
10	3	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
11	2	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
12	4	HD100014	COVED WAGSER 1" OD X 1/4" ID X 1/8" THICK
4A.5		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
4B.5		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
4C.5		AS104935	ANTI-TIP BENT TUBE LONG ASSEMBLY
6A-10		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
6B-10		AS103462	ANTI-TIP EXTENSION ASSY, LONG
1A,2A,3,4A,5,6A,7-12		AS117519	SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1B,2B,3,4A,5,6A,7-12		AS117520	SHORT ANTI-TIP W/ SHORT EXT, LEFT
1-3,4A,5,6A,7-12		AS117521	SHORT ANTI-TIP W/ SHORT EXT, (PAIR)
1A,2A,3,4A,5,6B,7-12		AS117522	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4A,5,6B,7-12		AS117523	SHORT ANTI-TIP W/ LONG EXT, LEFT
1-3,4A,5,6B,7-12		AS117524	SHORT ANTI-TIP W/ LONG EXT, (PAIR)
1A,2A,3,4B,5,6A,7-12		AS117525	ANTI-TIP W/ SHORT EXT, RIGHT
1B,2B,3,4B,5,6A,7-12		AS117526	ANTI-TIP W/ SHORT EXT, LEFT
1-3,4B,5,6A,7-12		AS117527	ANTI-TIP W/ SHORT EXT, (PAIR)

ASSEMBLIES ARE CONTINUED ON THE NEXT DAY

NEW USER FRIENDLY REAR FLIP-UP ANTI-TIP CONT.


TiLite™ Aero X, 2GX, Aero R, Aero X (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3,4B,5,6B,7-12		AS117528	ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4B,5,6B,7-12		AS117529	ANTI-TIP W/ LONG EXT, LEFT
1-3,4B,5,6B,7-12		AS117530	ANTI-TIP W/ LONG EXT, (PAIR)
1A,2A,3,4C,5,6A,7-12		AS117531	LONG ANTI-TIP W/ SHORT EXT, RIGHT
1B,2B,3,4C,5,6A,7-12		AS117532	LONG ANTI-TIP W/ SHORT EXT, LEFT
1-3,4C,5,6A,7-12		AS117533	LONG ANTI-TIP W/ SHORT EXT, (PAIR)
1A,2A,3,4C,5,6B,7-12		AS117534	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4C,5,6B,7-12		AS117535	LONG ANTI-TIP W/ LONG EXT, LEFT
1-3,4C,5,6B,7-12		AS117536	LONG ANTI-TIP W/ LONG EXT, (PAIR)



USER FRIENDLY REAR FLIP-UP ANTI-TIP

TiLite™ TRA, YRA

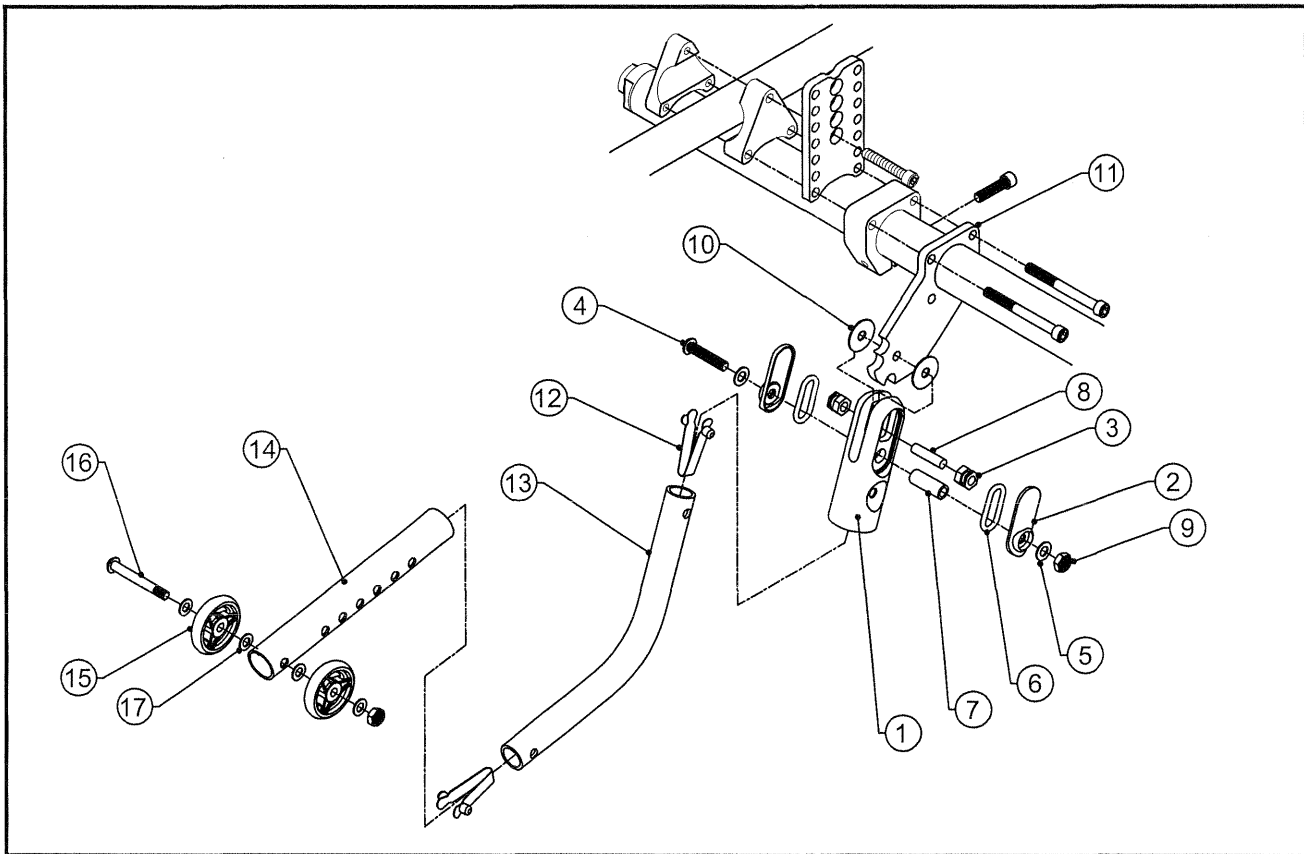


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP103135	ANTI-TIP PLATE
12	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
13A	1	CP103136	ANTI-TIP BENT TUBE SHORT
13B	1	CP103137	ANTI-TIP BENT TUBE
14A	1	CP100550	ANTI-TIP EXTENSION SHORT
14B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
15	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
16	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
17	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
1-11		AS103086	FLIP-UP ANTI-TIP SUB-ASSEMBLY
12-13A		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
12-13B		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
14A-17		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
14B-17		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-13A,14A-17		AS103145	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT
1-13A,14A-17		AS103149	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT PAIR
1-13A,14B-17		AS103146	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG
1-13A,14B-17		AS103150	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG PAIR
1-13B,14A-17		AS103147	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT
1-13B,14A-17		AS103151	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT PAIR
1-13B,14B-17		AS103148	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG
1-13B,14B-17		AS103152	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG PAIR




USER FRIENDLY REAR FLIP-UP ANTI-TIP

TiLite™ EVO, X, SX, YG, YGS

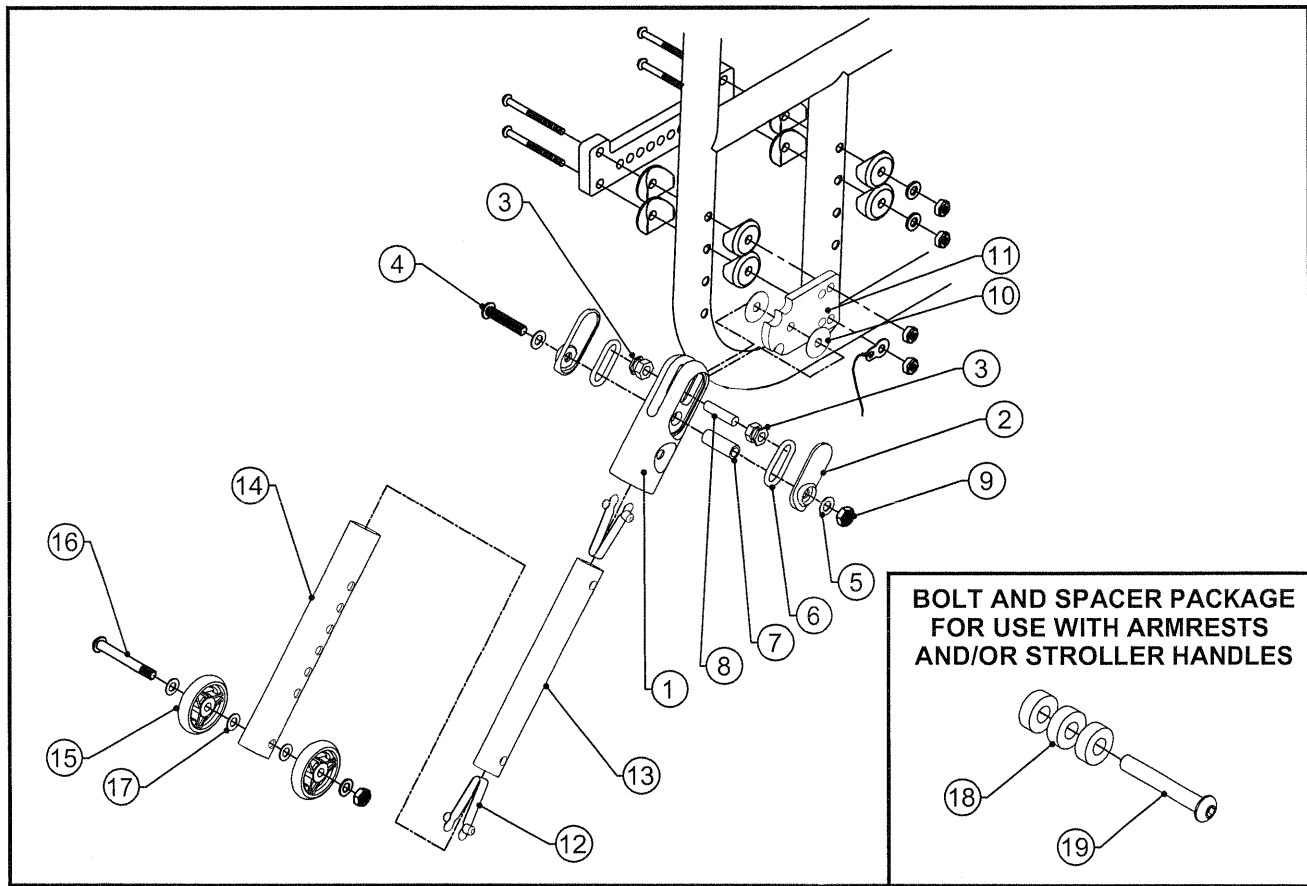


ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP102594	ANTI-TIP PLATE
12	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
13A	1	CP102891	ANTI-TIP STRAIGHT TUBE SHORT
13B	1	CP102892	ANTI-TIP STRAIGHT TUBE LONG
14A	1	CP100550	ANTI-TIP EXTENSION SHORT
14B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
15	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
16	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
17	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
18	12	CP102035	NYLON SPACER .25ID X .75OD X .25TH
19A	4	HD100245	1/4-28 X 2-1/2" BUTTON HEAD CAP SCREW
19B	4	HD100351	1/4-28 X 2-3/4" BUTTON HEAD CAP SCREW, ALLOY STEEL, CLEAR ZINC
1-11		AS103087	FLIP-UP ANTI-TIP SUB-ASSEMBLY
12-13A		AS103088	ANTI-TIP STRAIGHT TUBE SHORT ASSEMBLY
12-13A		AS103089	ANTI-TIP STRAIGHT TUBE LONG ASSEMBLY
14A-17		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
14B-17		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-13A,14A-17		AS103153	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT
1-13A,14A-17		AS103157	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT PAIR
1-13A,14B-17		AS103154	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG
1-13A,14B-17		AS103158	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG PAIR
1-13B,14A-17		AS103155	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT
1-13B,14A-17		AS103159	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT PAIR
1-13B,14B-17		AS103156	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG
1-13B,14B-17		AS103160	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG PAIR
18,19A,19B		AS103786	FLIP-UP ANTI-TIP ASSEMBLY SPACER PACK

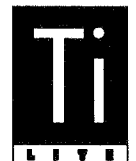

USER FRIENDLY REAR FLIP-UP ANTI-TIP

TiLite™ TX

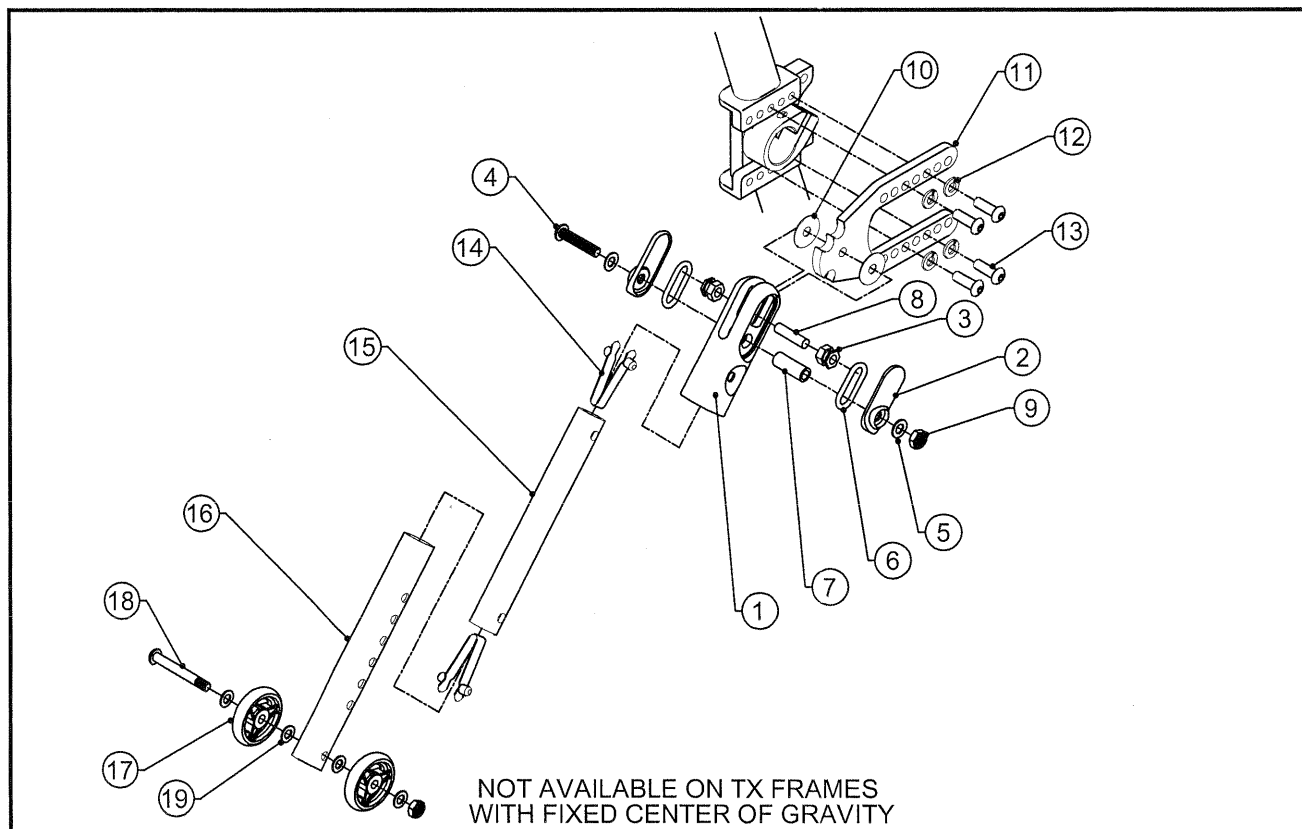


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP102536	ANTI-TIP ADAPTER PLATE
12	4	HD100156	1/4" LOCK WASHER
13	4	HD100074	1/4-28 X 5/8 BUTTON HEAD CAP SCREW
14	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
15A	1	CP102891	ANTI-TIP STRAIGHT TUBE SHORT
15B	1	CP102892	ANTI-TIP STRAIGHT TUBE LONG
16A	1	CP100550	ANTI-TIP EXTENSION SHORT
16B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
17	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
18	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
19	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THICK
1-13		AS103084	FLIP-UP ANTI-TIP SUB-ASSEMBLY
14-15A		AS103088	ANTI-TIP STRAIGHT TUBE SHORT ASSEMBLY
14-15B		AS103089	ANTI-TIP STRAIGHT TUBE LONG ASSEMBLY
16A-19		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
16B-19		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-15A, 16A-19		AS103090	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT
1-15A, 16A-19		AS103133	FLIP-UP ANTI-TIP ASSEMBLY SHORT-SHORT PAIR
1-15A, 16B-19		AS103091	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG
1-15A, 16B-19		AS103134	FLIP-UP ANTI-TIP ASSEMBLY SHORT-LONG PAIR
1-15B, 16A-19		AS103092	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT
1-15B, 16A-19		AS103135	FLIP-UP ANTI-TIP ASSEMBLY LONG-SHORT PAIR
1-15B, 16B-19		AS103093	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG
1-15B, 16B-19		AS103136	FLIP-UP ANTI-TIP ASSEMBLY LONG-LONG PAIR



REAR FLIP UP USER FRIENDLY ANTI-TIPS W/ AMPUTEE AXLE

TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

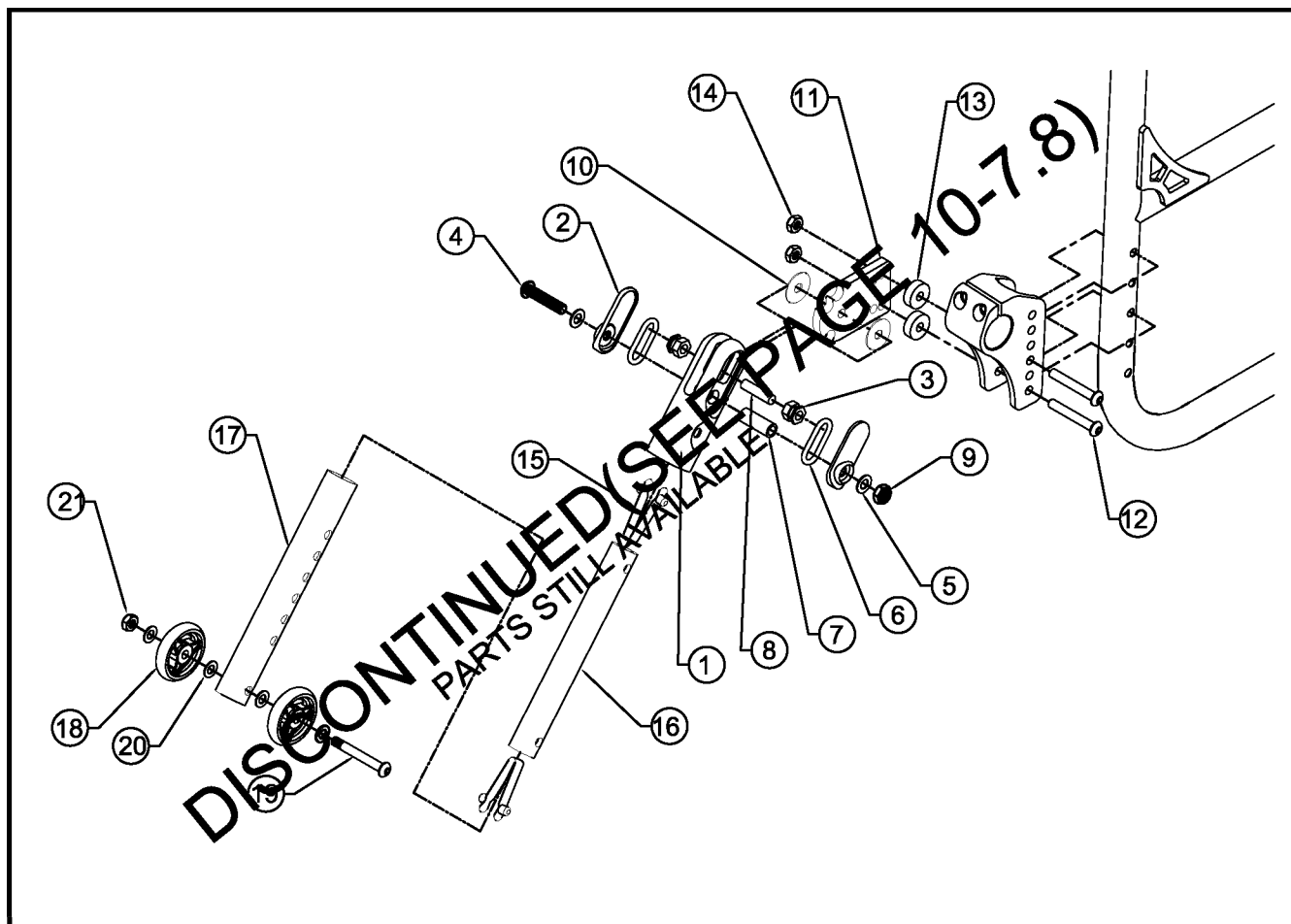


ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP103183	BILLET ANTI-TIP FORK
2	2	CP102847	ANTI-TIP FORK COVER
3	2	CP103202	ANTI-TIP HEX BUSHING
4	1	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THK
6	2	HD100410	O-RING, NEOPRENE 1/4 ID X 3/4 OD X 1/8 THK
7	1	HD100467	BRASS SLEEVE .375 OD X .25 ID X 1.00
8	1	HD100468	DOWEL PIN .25 OD X 1.00 LONG
9	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	2	HD100421	WASHER NYLON .253 ID X .878 OD X .031 THK
11	1	CP102594	ANTI-TIP PLATE
12	2	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
13	2	CP102035	NYLON SPACER .25ID X .75OD X .25TH
14	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
15	2	CP100251	SNAP BUTTON, .240 O.D. X .290 TALL
16A	1	CP102891	ANTI-TIP STRAIGHT TUBE SHORT
16B	1	CP102892	ANTI-TIP STRAIGHT TUBE LONG
17A	1	CP100550	ANTI-TIP EXTENSION SHORT
17B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
18	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
19	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
20	4	HD100067	NYLON WASHER, .500 O.D. X .265 I.D. X .031 THK
21	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
1-11		AS103087	FLIP-UP ANTI-TIP SUB-ASSEMBLY
15-16A		AS103088	ANTI-TIP STRAIGHT TUBE SHORT ASSEMBLY
15-16B		AS103089	ANTI-TIP STRAIGHT TUBE LONG ASSEMBLY
17A-21		AS103461	FLIP-UP ANTI-TIP EXTENSION ASSY, SHORT
17B-21		AS103462	FLIP-UP ANTI-TIP EXTENSION ASSY, LONG
1-16A, 17A-21		AS110367	FLIP-UP ANTI-TIP AMPUTEE ASSY, SHORT-SHORT
1-16A, 17A-21		AS110368	FLIP-UP ANTI-TIP AMPUTEE ASSY, SHORT-SHORT PAIR
1-16A, 17B-21		AS110369	FLIP-UP ANTI-TIP AMPUTEE ASSY, SHORT-LONG
1-16A, 17B-21		AS110370	FLIP-UP ANTI-TIP AMPUTEE ASSY, SHORT-LONG PAIR
1-16B, 17A-21		AS110371	FLIP-UP ANTI-TIP AMPUTEE ASSY, LONG-SHORT
1-16B, 17A-21		AS110372	FLIP-UP ANTI-TIP AMPUTEE ASSY, LONG-SHORT PAIR
1-16B, 17B-21		AS110373	FLIP-UP ANTI-TIP AMPUTEE ASSY, LONG-LONG
1-16B, 17B-21		AS110374	FLIP-UP ANTI-TIP AMPUTEE ASSY, LONG-LONG PAIR



NEW REAR FLIP UP USER FRIENDLY ANTI-TIPS W/ AMPUTEE AXLE

TiLite™ Aero X, 2GX, Aero X (SERIES 2)

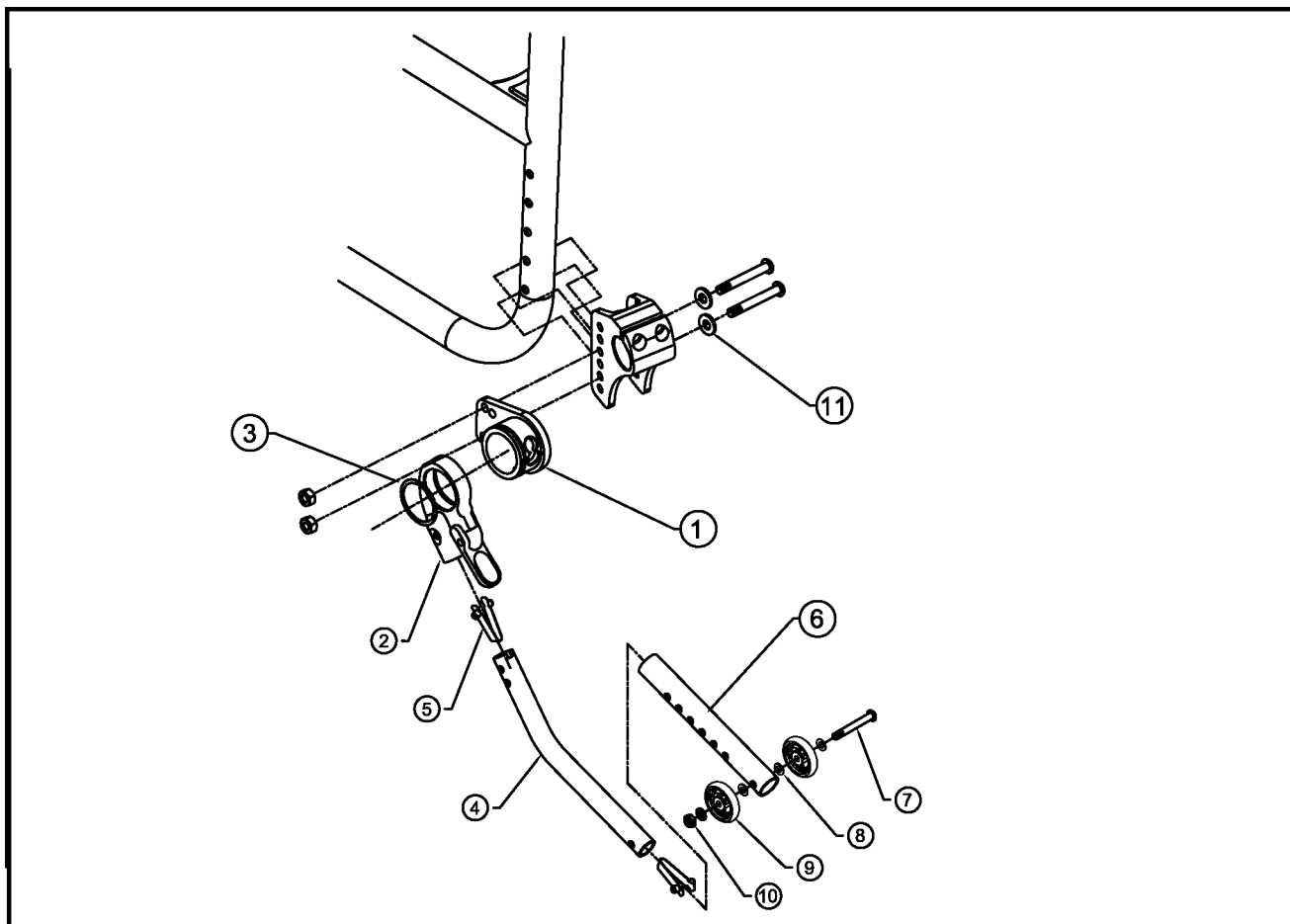

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3		AS117517	FOLDING UFAT SUB ASSEMBLY RIGHT
1B,2B,3		AS117518	FOLDING UFAT SUB ASSEMBLY LEFT
4A	1	CP103136	ANTI TIP BENT TUBE SHORT
4B	1	CP103137	ANTI TIP BENT TUBE
4C	1	CP104586	ANTI-TIP BENT TUBE, LONG
5	2	CP100251	SNAP BUTTON, .240 OD X .290 TALL
6A	1	CP100550	ANTI-TIPPER EXTENSION SHORT
6B	1	CP100212	ANTI-TIP LOWER EXTENSION TUBE
7	3	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
8	4	HD100067	NYLON WASHER, .500 OD X .265 ID X .031 THICK
9	2	CP103207	ANTI-TIP WHEEL, TRI-SPOKE
10	3	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
11	2	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
4A,5		AS103374	ANTI-TIP BENT TUBE SHORT ASSEMBLY
4B,5		AS103375	ANTI-TIP BENT TUBE ASSEMBLY
4C,5		AS104935	ANTI-TIP BENT TUBE LONG ASSEMBLY
6A-10		AS103461	ANTI-TIP EXTENSION ASSY, SHORT
6B-10		AS103462	ANTI-TIP EXTENSION ASSY, LONG
1A,2A,3,4A,5,6A,7-11		AS117537	SHORT ANTI-TIP W/ SHORT EXT, RIGHT
1B,2B,3,4A,5,6A,7-11		AS117538	SHORT ANTI-TIP W/ SHORT EXT, LEFT
1-3,4A,5,6A,7-11		AS117539	SHORT ANTI-TIP W/ SHORT EXT, (PAIR)
1A,2A,3,4A,5,6B,7-11		AS117540	SHORT ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4A,5,6B,7-11		AS117541	SHORT ANTI-TIP W/ LONG EXT, LEFT
1-3,4A,5,6B,7-11		AS117542	SHORT ANTI-TIP W/ LONG EXT, (PAIR)
1A,2A,3,4B,5,6A,7-11		AS117543	ANTI-TIP W/ SHORT EXT, RIGHT
1B,2B,3,4B,5,6A,7-11		AS117544	ANTI-TIP W/ SHORT EXT, LEFT
1-3,4B,5,6A,7-11		AS117545	ANTI-TIP W/ SHORT EXT, (PAIR)

NEW REAR FLIP UP USER FRIENDLY ANTI-TIPS W/ AMPUTEE AXLE CONT.


TiLite™ Aero X, 2GX, Aero R, Aero X (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3,4B,5,6B,7-11		AS117546	ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4B,5,6B,7-11		AS117547	ANTI-TIP W/ LONG EXT, LEFT
1-3,4B,5,6B,7-11		AS117548	ANTI-TIP W/ LONG EXT, (PAIR)
1A,2A,3,4C,5,6A,7-11		AS117549	LONG ANTI-TIP W/ SHORT EXT, RIGHT
1B,2B,3,4C,5,6A,7-11		AS117550	LONG ANTI-TIP W/ SHORT EXT, LEFT
1-3,4C,5,6A,7-11		AS117551	LONG ANTI-TIP W/ SHORT EXT, (PAIR)
1A,2A,3,4C,5,6B,7-11		AS117552	LONG ANTI-TIP W/ LONG EXT, RIGHT
1B,2B,3,4C,5,6B,7-11		AS117553	LONG ANTI-TIP W/ LONG EXT, LEFT
1-3,4C,5,6B,7-11		AS117554	LONG ANTI-TIP W/ LONG EXT, (PAIR)



FIFTH WHEEL ANTI-TIP

TR (SERIES 3)

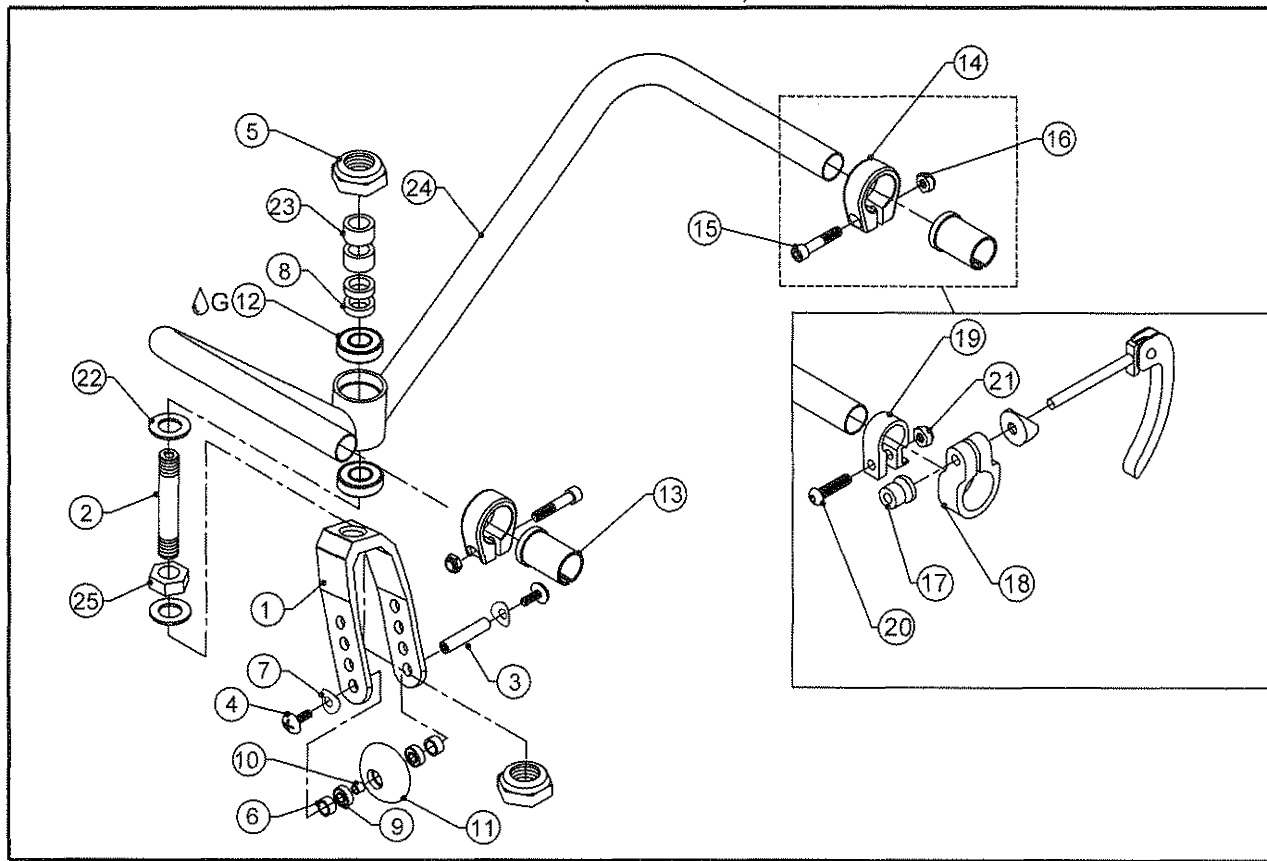


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP107307	NEW 5TH WHEEL FORK
2	1	CP101415	CASTER STEM STD. W/ 1/2" THREAD 1" EXTENSION
3	1	HD100085	5"-8" CASTER AXLE
4	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
5	2	HD100087	1/2-20 UNF LCK NUT THIN
6	2	CP103006	.316ID X .438OD X .317L CASTER AXLE SPACER
7	2	CP104375	C SPRING WASHER
8	2	CP100937	ANTI-TIP SPACER .200 THICK
9	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W
10	1	CP100006	.438 X .400 SPACER
11	1	CP100199	HYPER X-360-82mm, 82A
12 Δ G	2	HD100011	BEARING 1/2" I.D. X 1-1/8" O.D. X 5-16" W
13	2	CP100934	SLEEVE 1 X .051 TO 3/4
14	2	CP106242	ONE INCH TUBE FOOTREST CLAMP
15	2	HD100050	1/4-20 X 3/4" SOCKET HEAD CAP SCREW
16	2	HD100049	1/4-20 NYLOCK LOCKNUT THIN STYLE
17	2	HD100235	ATB BINDER BOLT
18	2	CP100921	ANTI-TIP CLAMP, BB CHAIR
19	2	CP100935	CLAMP, BINDER 3/4
20	2	HD100065	1/4-20 X 1" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
21	2	HD100049	1/4-20 NYLOCK LOCKNUT THIN STYLE
22	2	HD100007	1/2 AN WASHER .500ID .875OD .06
23	2	CP100346	CASTER EXTENSION 1/2"
CON. 24	1	CP101168	FIFTH WHEEL ANTI-TIP CUSTOM
25	1	HD100086	1/2-20 UNF JAM NUT
9-11		AS100356	3" ROLLER BLADE WHEEL ASSY
1-16, 22-25		AS100597	FIFTH WHEEL ANTI-TIP STD
1-13, 17-25		AS100560	FIFTH WHEEL ANTI-TIP (QUICK RELEASE)

CON.= CONSULT FACTORY FOR SIZES WHEN ORDERING

CENTER MOUNT SINGLE SWIVEL CASTER ANTI-TIP ASSEMBLY

TiLite™ TWIST

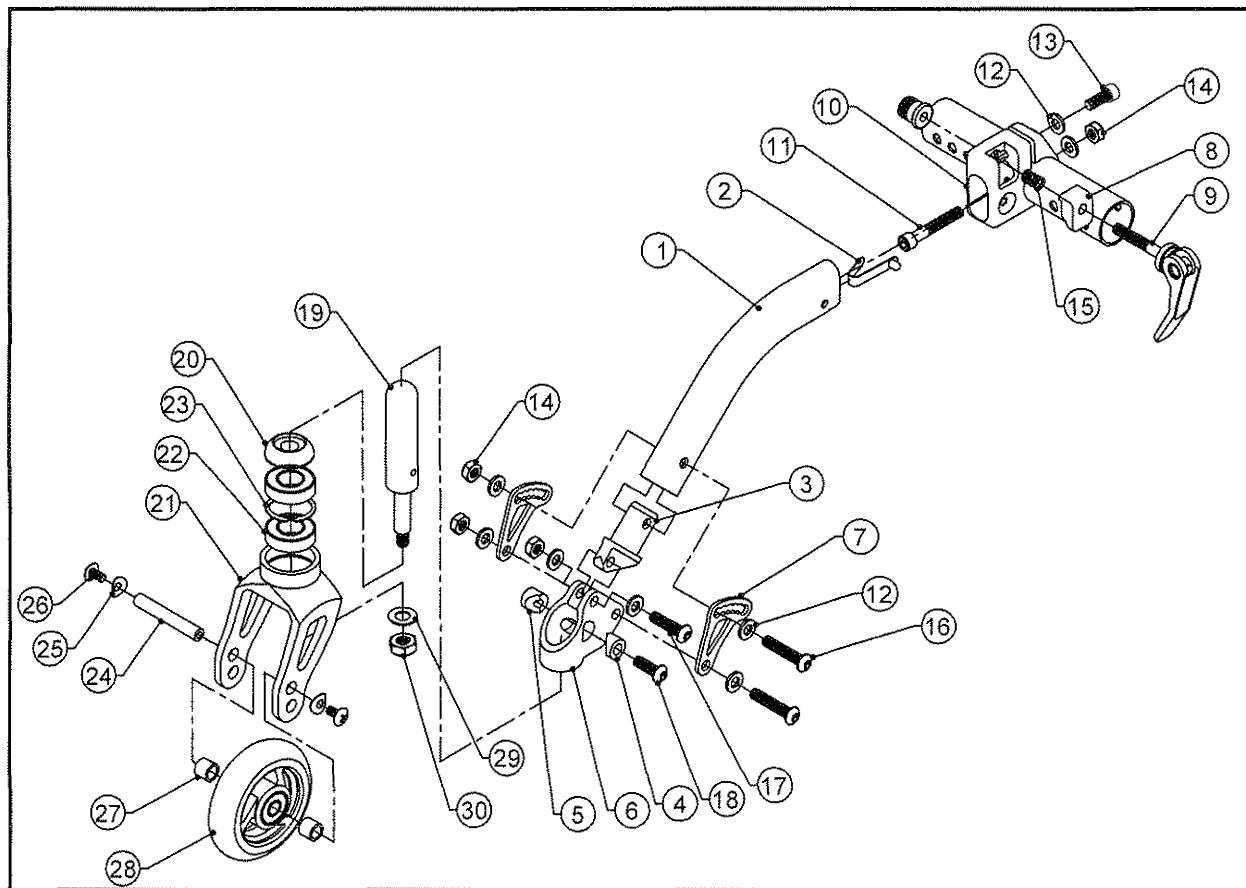

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	ANTI-TIP STRUT, SWIVEL
2	1	HD100673	SINGLE END SNAP BUTTON, Ø.188
3	1	CP105753	PRESS-IN TUBE END
4	1	CP105749	BULLET BARREL CLAMP "A"
5	1	CP105750	BULLET BARREL CLAMP "B"
6	1	CP105738	ANTI-TIP BULLET CLAMP
7	2	CP105748	ANGLE ADJUSTMENT CAM
8	1	CP105752	CENTER MOUNT ANTI-TIP RECIEVER CLAMP
9	1	HD100674	QUICK RELEASE CAM LOCK, M6 X 55mm
10	1	CP105751	CENTER MOUNT ANTI-TIP RECIEVER
11	1	HD100563	1/4-28 X 1-3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
12	8	HD100490	1/4" AN WASHER, .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
13	1	HD100458	1/4-20 X 5/8" SOCKET HEAD CAP SCREW, BLACK ZINC
14	4	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
15	1	HD100429	SPRING, .328 OD. X .278 ID. X .630 LONG
16	2	HD100438	1/4-28 X 1-1/4" BUTTON HEAD CAP SCREW, BLACK
17	1	HD100512	1/4-28 X 1" BUTTON HEAD CAP SCREW, BLACK ZINC
18	1	HD100646	1/4-20 X 3/4" BUTTON HEAD CAP SCREW, BLACK ZINC
19	1	AS111723	SWIVEL CASTER BULLET W/ STEM
20	1	CP103744	CASTER CAP, MONO FORK
21	1	CP103567	WIDE BEARING FORK SHORT
22	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
23	1	HD100330	1-1/8" INTERNAL RETAINING RING
24	1	CP103041	CASTER AXLE LONG
25	2	CP104375	C SPRING WASHER
26	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
27	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
28	1	AS100356	3" MICRO CASTER ASSY
29	1	HD100353	3/8" AN WASHER STAINLESS STEEL
30	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT

CENTER MOUNT SINGLE SWIVEL CASTER ANTI-TIP ASSEMBLY (CONTINUED)

TiLite™ TWIST

ITEM	QUANTITY	PART NO.	DESCRIPTION
1-2		TABLE TWO	ANTI-TIP STRUT, SWIVEL ASSY
19-30		AS111724	SWIVEL CASTER ASSEMBLY
1-30		TABLE THREE	CENTER MOUNT ANTI-TIP, SWIVEL ASSEMBLY
8-15		AS116025	PRESS-IN TUBE END ASSEMBLY
3-7, 12, 14, 16-18		AS116096	ANGLE ADJUSTMENT MOUNT

TABLE ONE - ANTI-TIP STRUT, SWIVEL

REAR WHEEL	PART NUMBER	DESCRIPTION
18"	CP105740	ANTI-TIP STRUT, Ø18" WHEEL, SWIVEL
20"	CP105742	ANTI-TIP STRUT, Ø20" WHEEL, SWIVEL
22"	CP105744	ANTI-TIP STRUT, Ø22" WHEEL, SWIVEL
24"	CP105746	ANTI-TIP STRUT, Ø24" WHEEL, SWIVEL

TABLE TWO - ANTI-TIP STRUT, SWIVEL ASSEMBLY

REAR WHEEL	ASSEMBLY NUMBER	DESCRIPTION
18"	AS111719	ANTI-TIP STRUT, Ø18" WHEEL, SWIVEL ASSY
20"	AS111720	ANTI-TIP STRUT, Ø20" WHEEL, SWIVEL ASSY
22"	AS111721	ANTI-TIP STRUT, Ø22" WHEEL, SWIVEL ASSY
24"	AS111722	ANTI-TIP STRUT, Ø24" WHEEL, SWIVEL ASSY


TABLE THREE - CENTER MOUNT ANTI-TIP, SWIVEL ASSEMBLY

REAR WHEEL	ASSEMBLY NUMBER	DESCRIPTION
18"	AS111725	CENTER MOUNT ANTI-TIP, Ø18" WHEEL, SWIVEL ASSY
20"	AS111726	CENTER MOUNT ANTI-TIP, Ø20" WHEEL, SWIVEL ASSY
22"	AS111727	CENTER MOUNT ANTI-TIP, Ø22" WHEEL, SWIVEL ASSY
24"	AS111728	CENTER MOUNT ANTI-TIP, Ø24" WHEEL, SWIVEL ASSY



CENTER MOUNT SINGLE NON-SWIVEL ANTI-TIP ASSEMBLY

TiLite™ TWIST

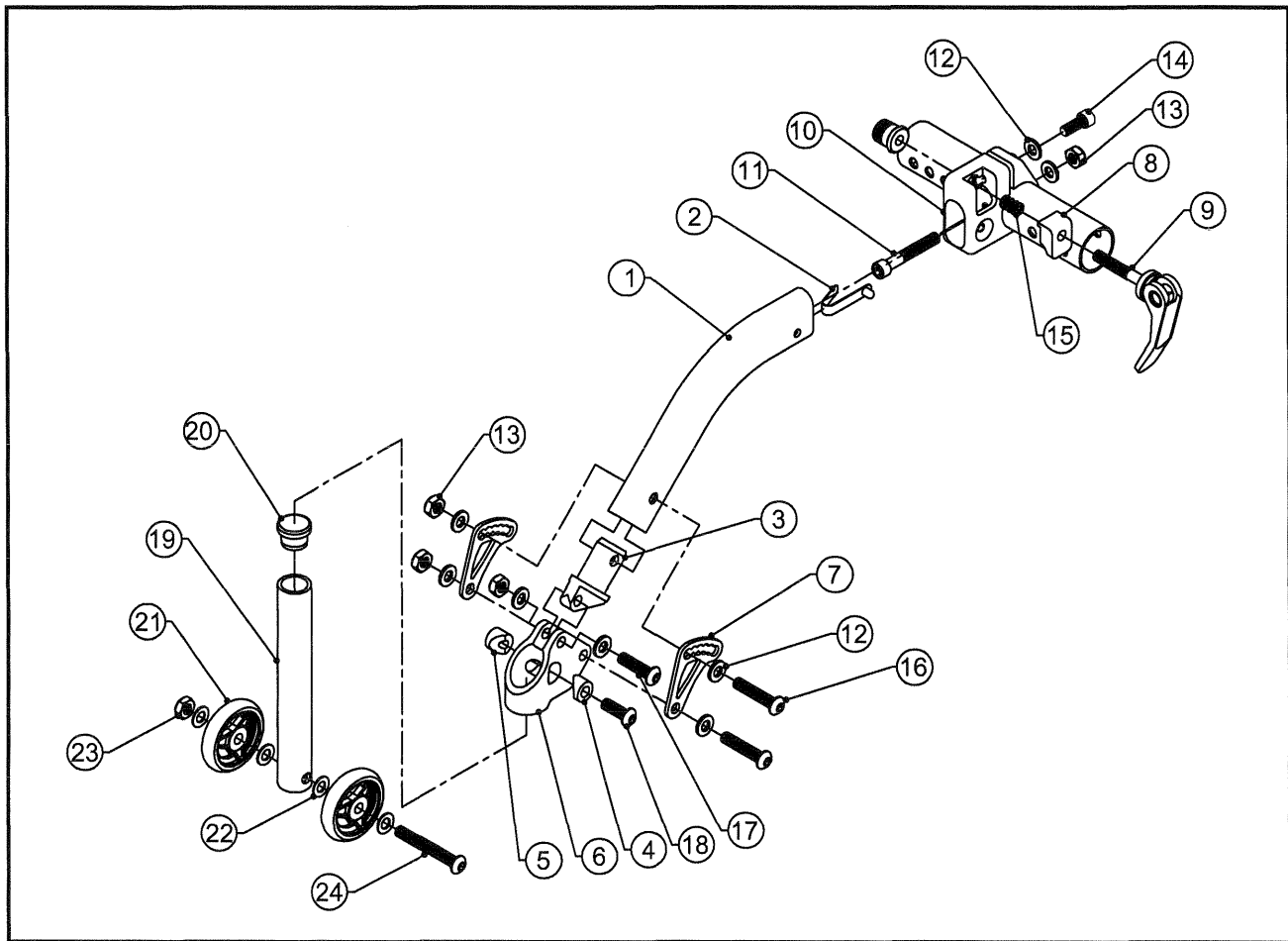

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	ANTI-TIP STRUT, FIXED
2	1	HD100673	SINGLE END SNAP BUTTON, Ø.188
3	1	CP105753	PRESS-IN TUBE END
4	1	CP105749	BULLET BARREL CLAMP "A"
5	1	CP105750	BULLET BARREL CLAMP "B"
6	1	CP105738	ANTI-TIP BULLET CLAMP
7	2	CP105748	ANGLE ADJUSTMENT CAM
8	1	CP105752	CENTER MOUNT ANTI-TIP RECIEVER CLAMP
9	1	HD100674	QUICK RELEASE CAM LOCK, M6 X 55mm
10	1	CP105751	CENTER MOUNT ANTI-TIP RECIEVER
11	1	HD100563	1/4-28 X 1-3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
12	8	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
13	4	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
14	1	HD100458	1/4-20 X 5/8" SOCKET HEAD CAP SCREW, BLACK ZINC
15	1	HD100429	SPRING, .328 OD. X .278 ID. X .630 LONG
16	2	HD100438	1/4-28 X 1-1/4" BUTTON HEAD CAP SCREW, BLACK
17	1	HD100512	1/4-28 X 1" BUTTON HEAD CAP SCREW, BLACK ZINC
18	1	HD100646	1/4-20 X 3/4" BUTTON HEAD CAP SCREW, BLACK ZINC
19	1	CP105754	FIXED ANTI-TIP TUBE
20	1	HD100127	END CAP 3/4" X .083 WALL TUBING
21	2	CP103207	ANTI-TIP WHEEL TRI-SPOKE
22	4	HD100067	NYLON WASHER .500 OD X .265 ID X .031
23	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
24	1	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK

CONTINUED ON NEXT PAGE

CENTER MOUNT SINGLE NON-SWIVEL ANTI-TIP ASSEMBLY (CONTINUED)

TiLite™ TWIST

ITEM	QUANTITY	PART NO.	DESCRIPTION
1-2		TABLE TWO	ANTI-TIP STRUT, FIXED ASSY
18-23		AS111729	NON-SWIVEL CASTER ASSEMBLY
1-23		TABLE THREE	CENTER MOUNT ANTI-TIP, NON-SWIVEL ASSEMBLY
8-15		AS116025	PRESS-IN TUBE END ASSEMBLY
3,-7,12,13,16-18		AS116096	ANGLE ADJUSTMENT MOUNT

TABLE ONE - ANTI-TIP STRUT, FIXED

REAR WHEEL	PART NUMBER	DESCRIPTION
18"	CP105741	ANTI-TIP STRUT, Ø18" WHEEL, FIXED
20"	CP105743	ANTI-TIP STRUT, Ø20" WHEEL, FIXED
22"	CP105745	ANTI-TIP STRUT, Ø22" WHEEL, FIXED
24"	CP105747	ANTI-TIP STRUT, Ø24" WHEEL, FIXED

TABLE TWO - ANTI-TIP STRUT, FIXED ASSEMBLY

REAR WHEEL	ASSEMBLY NUMBER	DESCRIPTION
18"	AS111730	ANTI-TIP STRUT, Ø18" WHEEL, FIXED ASSY
20"	AS111731	ANTI-TIP STRUT, Ø20" WHEEL, FIXED ASSY
22"	AS111732	ANTI-TIP STRUT, Ø22" WHEEL, FIXED ASSY
24"	AS111733	ANTI-TIP STRUT, Ø24" WHEEL, FIXED ASSY


TABLE THREE - CENTER MOUNT ANTI-TIP, NON-SWIVEL ASSY

REAR WHEEL	ASSEMBLY NUMBER	DESCRIPTION
18"	AS111734	CENTER MOUNT ANTI-TIP, Ø18" WHEEL, NON-SWIVEL ASSY
20"	AS111735	CENTER MOUNT ANTI-TIP, Ø20" WHEEL, NON-SWIVEL ASSY
22"	AS111736	CENTER MOUNT ANTI-TIP, Ø22" WHEEL, NON-SWIVEL ASSY
24"	AS111737	CENTER MOUNT ANTI-TIP, Ø24" WHEEL, NON-SWIVEL ASSY



TIP ASSIST


TiLite™ Aero X, AERO X2, 2GX, 2GX2, Aero R

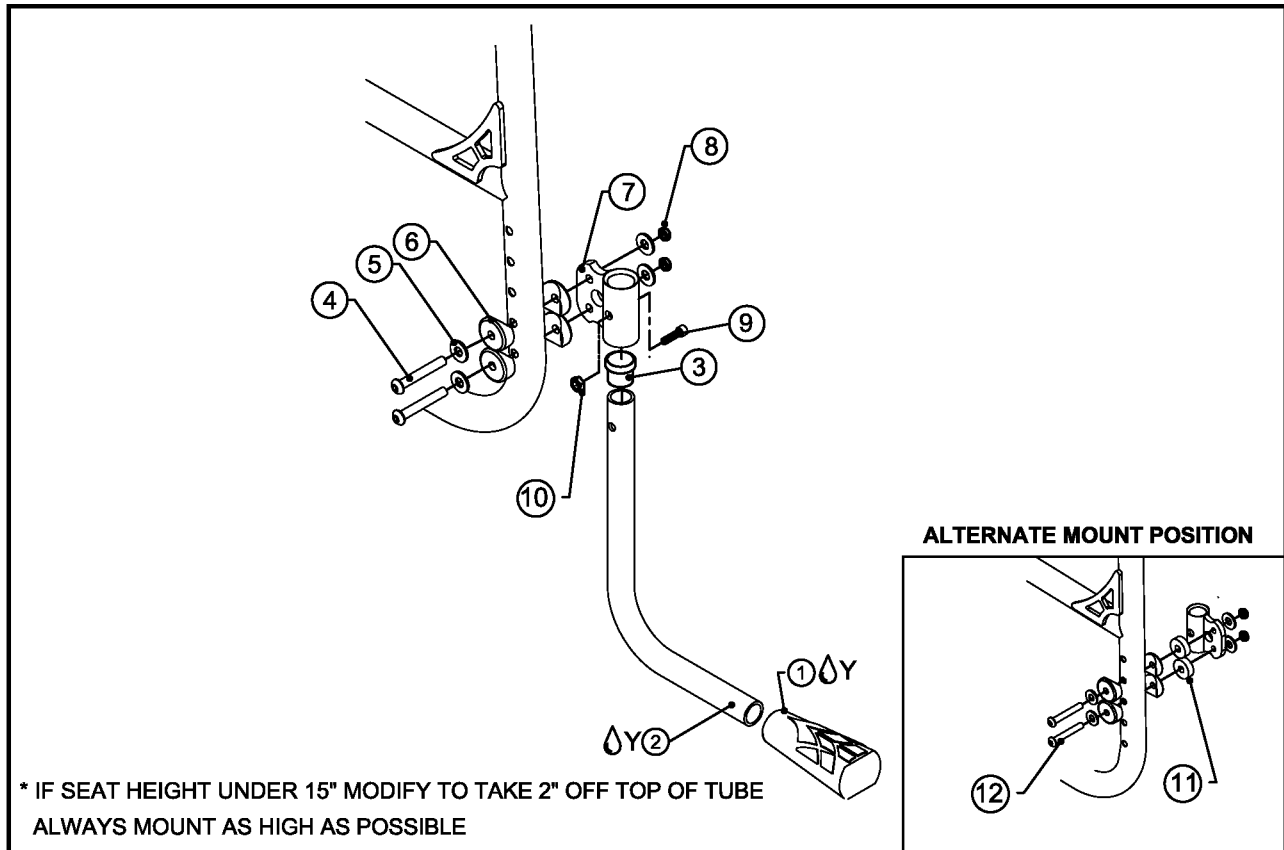

ITEM	QUANTITY	PART NO.	DESCRIPTION
1Y	1	CP107631	TIP ASSIST GRIP
2Y	1	CP107633	TIP ASSIST TUBE*
3	1	HD100127	END CAP 3/4" X .083 WALL TUBING
4	2	CP103096	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, CUSTOM THREAD LENGTH, BLACK
5	4	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
6	4	HD100014	COVED WASHER 1" OD X 1/4" ID X 1/8" THK
7	1	CP107632	TIP ASSIST BRACKET
8	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
9	1	HD100562	1/4-20 X 1-1/4" SOCKET HEAD CAP SCREW, BLACK ZINC
10	1	HD100469	1/4-20 NYLOCK LOCKNUT THIN STYLE, BLACK ZINC PLATE
11	2	CP102035	NYLON SPACER .25ID X .75OD X .25TH
12	2	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
1-10		AS117591	TIP ASSIST
1-3,5-12		AS117555	TIP ASSIST ALTERNATIVE MOUNT

TRAVEL AID ASSEMBLY


TiLite™ TR, TRC, YR, TR(SERIES 2), YR(SERIES 2), TR (SERIES 3)

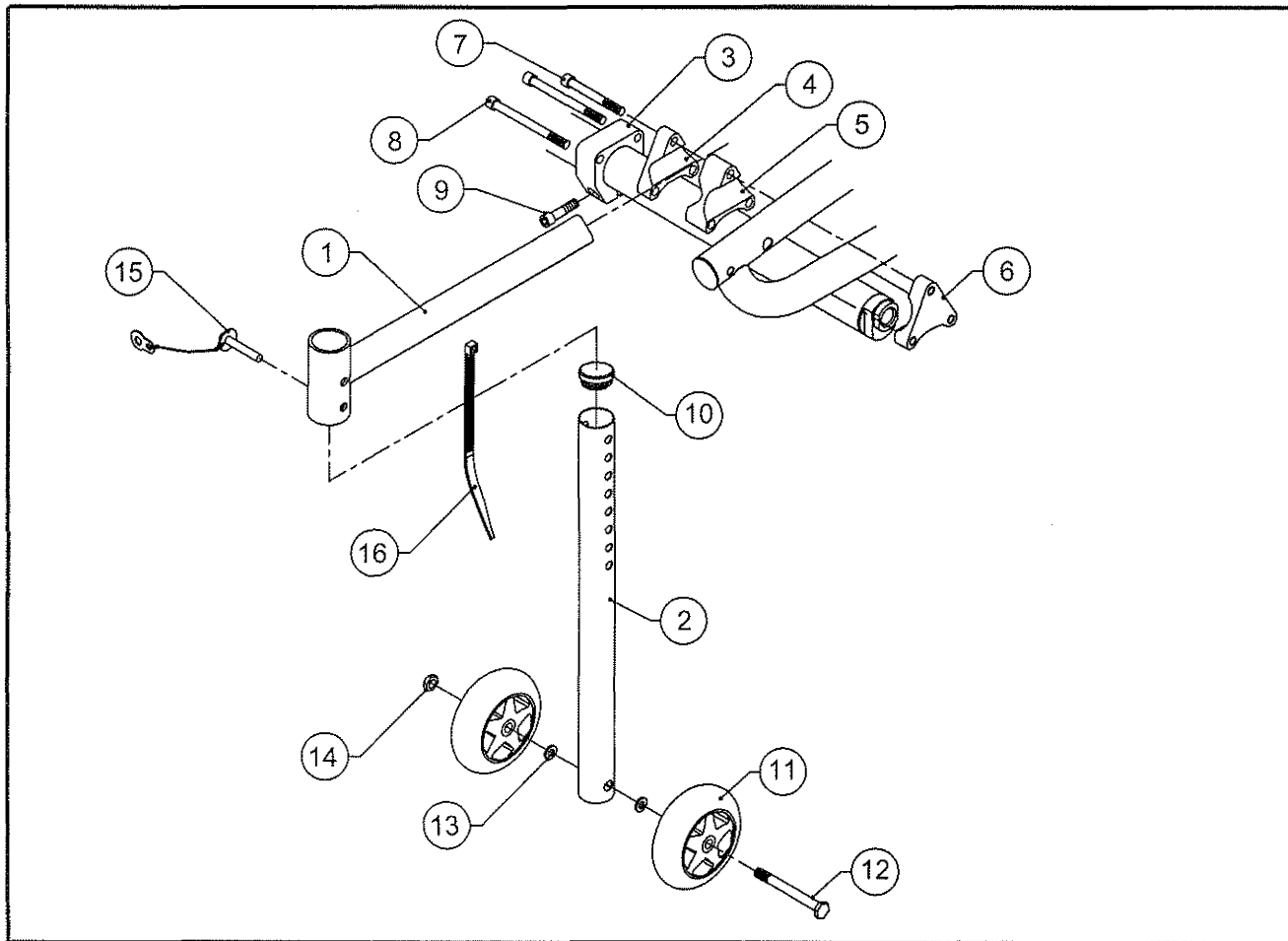

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101988	TRAVEL AID RECEPTACLE SQUARE
2	2	CP101989	TRAVEL AID TUBE
3	2	CP101063	ADJUSTABLE HEIGHT CAMBER TUBE MOUNT
4	2	CP101986	TRAVEL AID CLAMP SQUARE
5	2	CP101987	TRAVEL AID CLAMP SQUARE TO 1" TUBE
6	2	CP101061	CAMBER MOUNT THREADED
7	2	HD100227	1/4-20 X 2-1/2" SOCKET HEAD CAP SCREW
8	4	HD100322	1/4-20 X 3-1/2" SOCKET HEAD CAP SCREW
9	2	HD100051	1/4-20 X 1" SOCKET HEAD CAP SCREW
10	2	HD100134	END CAP 1" X .083 WALL TUBING
11	4	AS100356	3" ROLLER BLADE WHEEL ASSY
12	2	HD100325	5/16-24 X 3-5/16 AN SCREW
13	4	HD100098	5/16" AN WASHER .312ID .562OD .06 THICK
14	2	HD100107	5/16-24 NYLOCK LOCKNUT, THIN STYLE
15	2	AS100946	QR PIN STROLLER ASSY TR A
16	2	RW100057	CABLE TIE 8" LENGTH
1-16		AS102152	TRAVEL AID ASSEMBLY PAIR

TRAVEL AID ASSEMBLY


TiLite™ Aero X, 2GX, Aero R

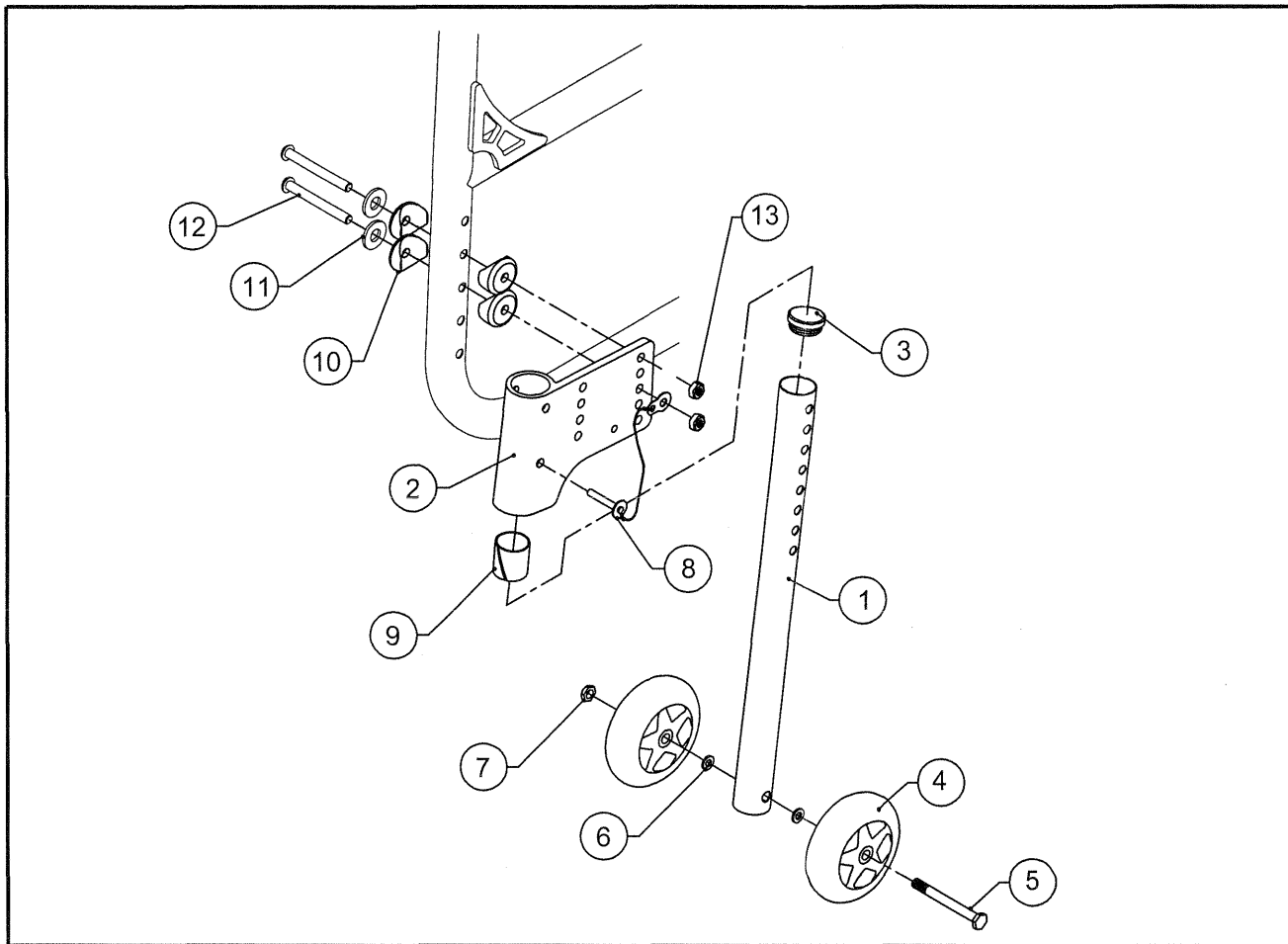

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101989	TRAVEL AID TUBE
2A	1	CP101203	RECEPTACLE STROLLER HANDLE RIGHT
2B	1	CP101204	RECEPTACLE STROLLER HANDLE LEFT
3	2	HD100134	END CAP 1" X .083 WALL TUBING
4	4	AS100356	3" ROLLER BLADE WHEEL ASSY
5	2	HD100276	5/16-24 X 3 AN SCREW
6	4	HD100098	5/16" AN WASHER .312ID .562OD .06 THICK
7	2	HD100107	5/16-24 NYLOCK LOCKNUT, THIN STYLE
8	2	AS100946	QR PIN STROLLER ASSY TR A
9	2	HD100222	NYLINER
10	8	HD100014	COVED WASHER 1" OD X 1/4" ID X 1/8" THK
11	4	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
12	4	HD100439	1/4-28 X 2" BUTTON HEAD CAP SCREW, BLACK
13	4	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
1-13		AS103590	TRAVEL AID ASSEMBLY PAIR

TRAVEL AID ASSEMBLY


TiLite™ TRA, YRA

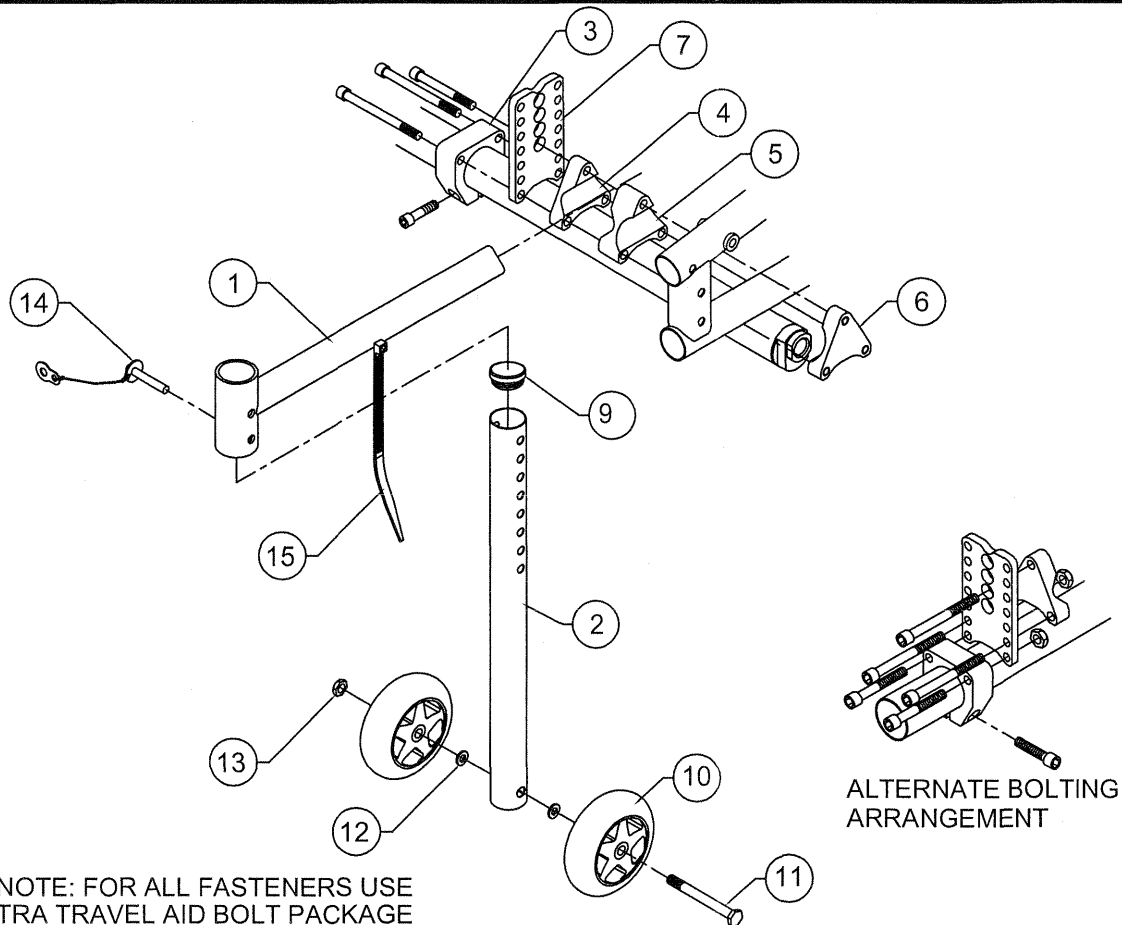

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101988	TRAVEL AID RECEPTACLE SQUARE
2	2	CP101989	TRAVEL AID TUBE
3	2	CP101063	ADJUSTABLE HEIGHT CAMBER TUBE MOUNT
4	2	CP101986	TRAVEL AID CLAMP SQUARE
5	2	CP101987	TRAVEL AID CLAMP SQUARE TO 1" TUBE
6	2	CP101061	CAMBER MOUNT THREADED
7	2	CP101062	ADJ HEIGHT CAMBER TUBE PLATE
8	4	AS102204	TRAVEL AID BOLT PACKAGE
9	2	HD100134	END CAP 1" X .083 WALL TUBING
10	4	AS100356	3" ROLLER BLADE WHEEL ASSY
11	2	HD100276	5/16-24 X 3 AN SCREW
12	4	HD100098	5/16" AN WASHER .312ID .562OD .06 THICK
13	2	HD100107	5/16-24 NYLOCK LOCKNUT, THIN STYLE
14	2	AS100946	QR PIN STROLLER ASSY
15	2	RW100057	CABLE TIE 8" LENGTH
1-15		AS102205	TRAVEL AID ASSEMBLY PAIR

TRAVEL AID ASSEMBLY


TiLite™ X, SX, EVO

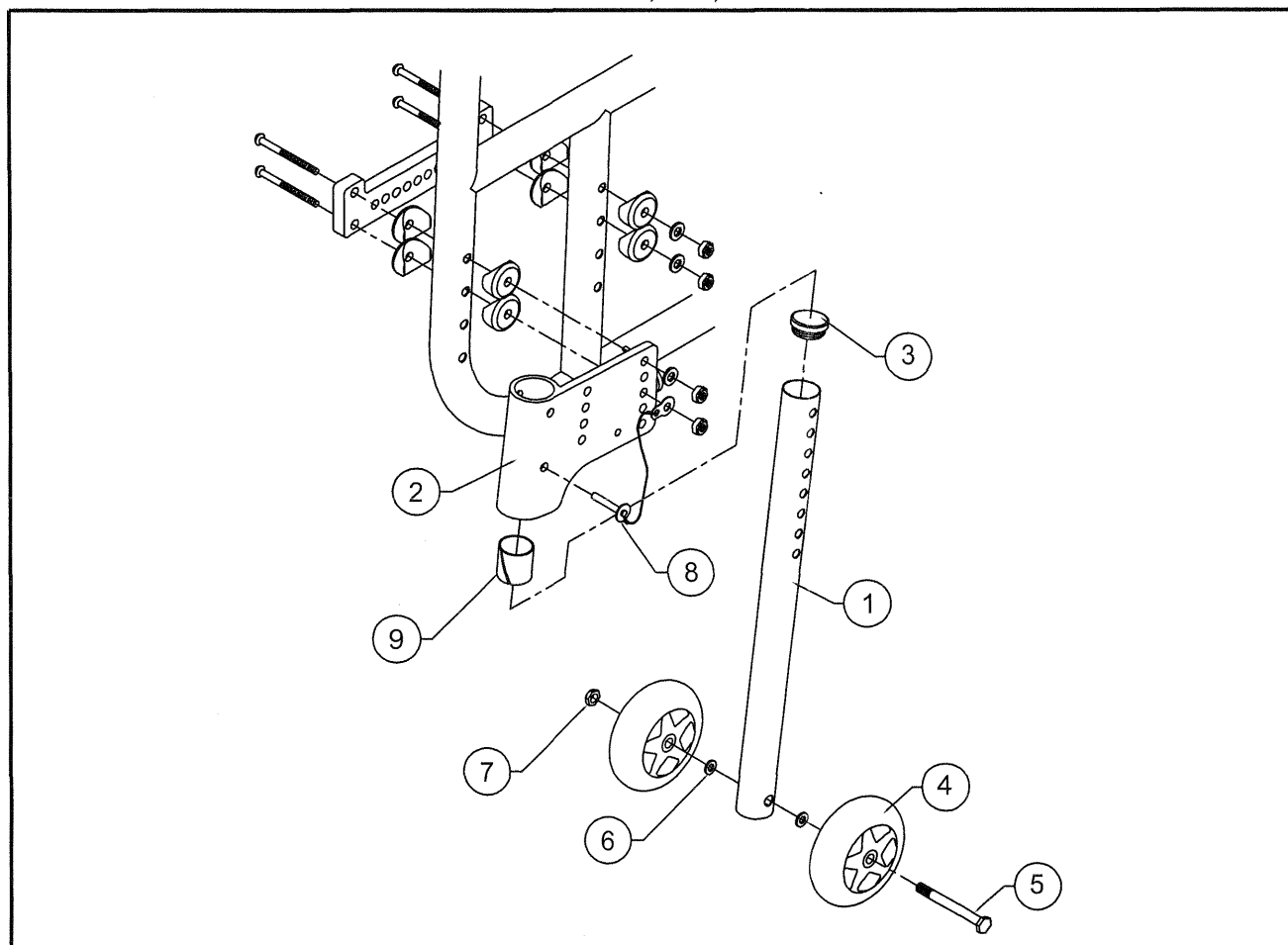

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101989	TRAVEL AID TUBE
2A	1	CP101203	RECEPTACLE STROLLER HANDLE RIGHT
2B	1	CP101204	RECEPTACLE STROLLER HANDLE LEFT
3	2	HD100134	END CAP 1" X .083 WALL TUBING
4	4	AS100356	3" ROLLER BLADE WHEEL ASSY
5	2	HD100276	5/16-24 X 3 AN SCREW
6	4	HD100098	5/16" AN WASHER .312ID .562OD .06 THICK
7	2	HD100107	5/16-24 NYLOCK LOCKNUT, THIN STYLE
8	2	AS100946	QR PIN STROLLER ASSY TR A
9	2	HD100222	NYLINER
1-9		AS102189	TRAVEL AID ASSEMBLY PAIR

ULTRASUEDE IMPACT GUARDS

TiLite™ TR, TRA, Evo, XC, X, SX, TX, Aero R, TR (SERIES 2), YR (SERIES 2),
TR (SERIES 3), AERO T, TRA (SERIES 2)

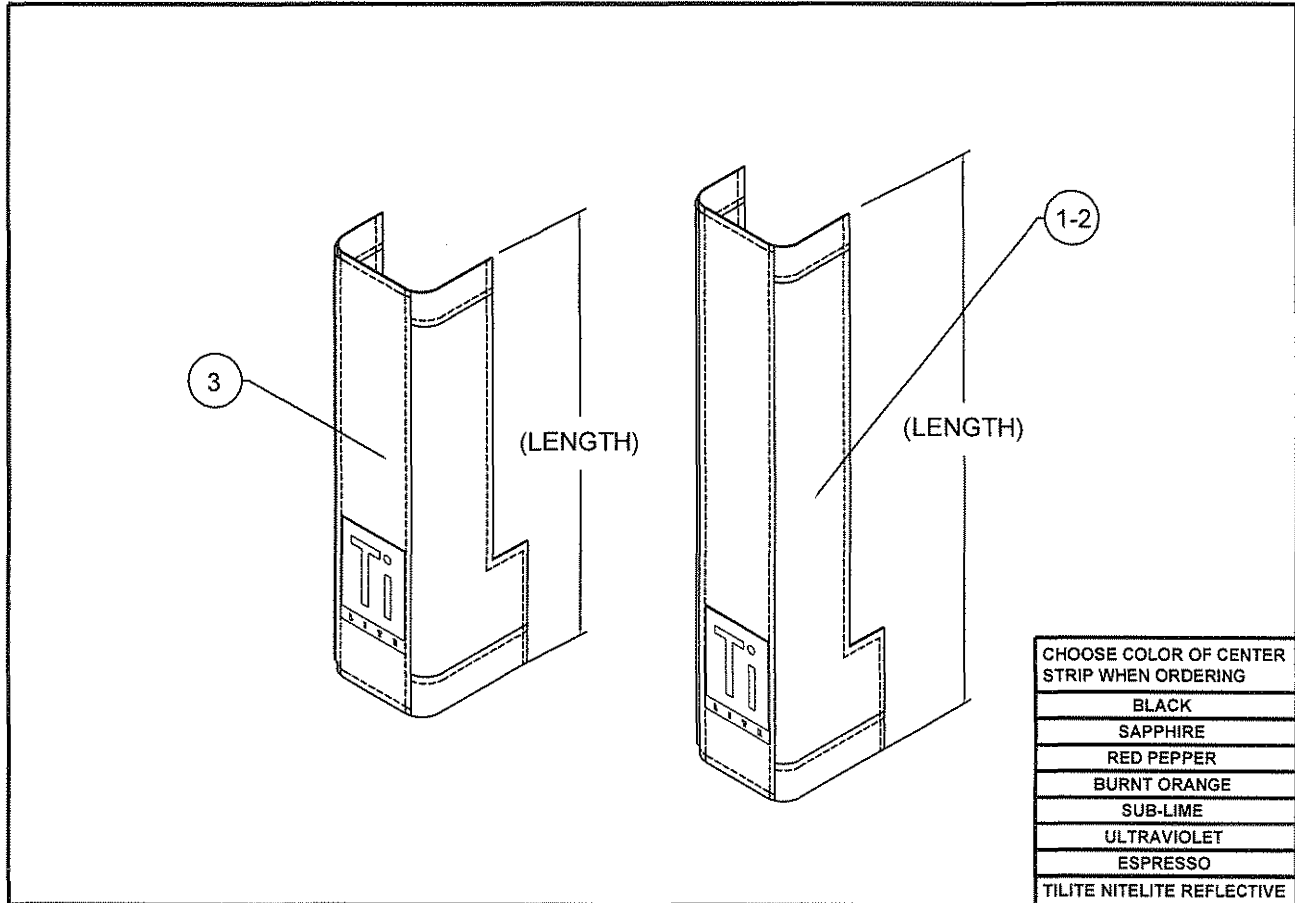


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP103480	ULTRASUEDE IMPACT GUARDS STD RIGHT-TR, TRC, TRA, YG, EVO, TX, AERO R (11.00")
1B	1	CP103481	ULTRASUEDE IMPACT GUARDS STD LEFT-TR, TRC, TRA, YG, EVO, TX, AERO R (11.00")
2	1	CP103482	ULTRASUEDE IMPACT GUARDS SHORT-TR, TRC, XC, X, SX, YGS, YG, EVO, TX, AERO R (8.00")
3A	1	CP103483	ULTRASUEDE IMPACT GUARDS RIGHT-SX, YGS (10.50")
3B	1	CP103484	ULTRASUEDE IMPACT GUARDS LEFT-SX, YGS (10.50")



NEOPRENE IMPACT GUARDS

TiLite™ ZR, ZRA, AERO Z, Z10

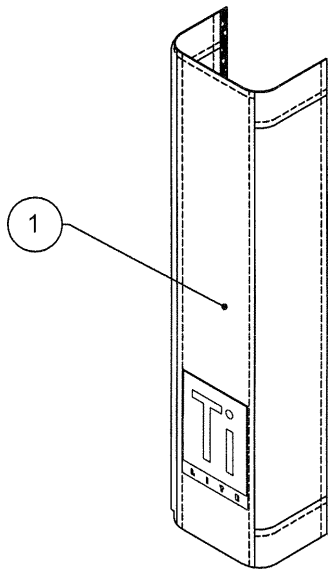


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103485	1.25" TUBING NEOPRENE IMPACT GUARD (10.00")
1B	1	CP106225	1.25" TUBING NEOPRENE IMPACT GUARD, SHORT (8.00")
1C	1	CP106226	1.25" TUBING NEOPRENE IMPACT GUARD, EXTRA SHORT (6.00")



ULTRASUEDE IMPACT GUARDS

TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

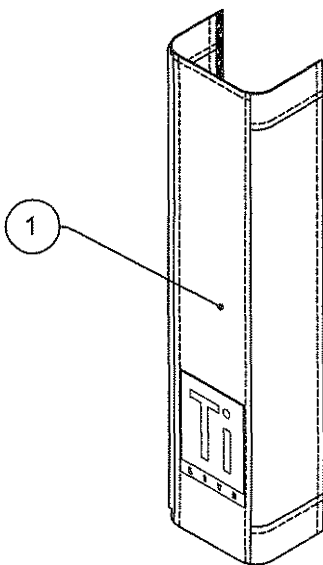

CHOOSE COLOR OF CENTER
STRIP WHEN ORDERING

BLACK

SAPPHIRE

RED PEPPER

BURNT ORANGE

SUB-LIME

ULTRAVIOLET

ESPRESSO


TILITE NITELITE REFLECTIVE

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104723	1.25" TUBING ULTRASUEDE IMPACT GUARD (10.00")
1B	1	CP105933	1.25" TUBING ULTRASUEDE IMPACT GUARD, SHORT (8.00")
1C	1	CP106224	1.25" TUBING ULTRASUEDE IMPACT GUARD, EXTRA SHORT (6.00")



ULTRASUEDE IMPACT GUARDS

TiLite™ TWIST

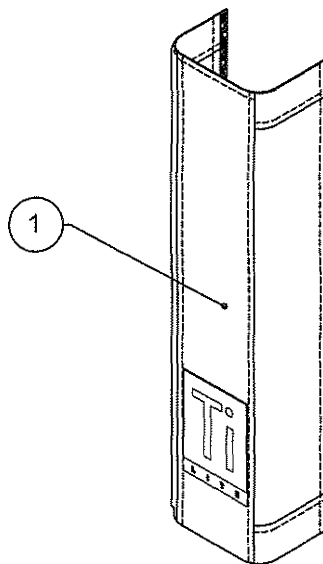


CHOOSE COLOR OF CENTER STRIP WHEN ORDERING
BLACK
SAPPHIRE
RED PEPPER
BURNT ORANGE
SUB-LIME
ULTRAVIOLET
ESPRESSO
TILITE NITELITE REFLECTIVE

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103482	ULTRASUEDE IMPACT GUARD, SHORT (8.00")
1B	1	CP105844	ULTRASUEDE IMPACT GUARD, EXTRA SHORT (6.00")



NEOPRENE IMPACT GUARDS


TiLite™ 2GX, Aero X, Aero R WITH SWINGAWAY FRAMES,
Aero X (SERIES 2), 2GX (SERIES 2)

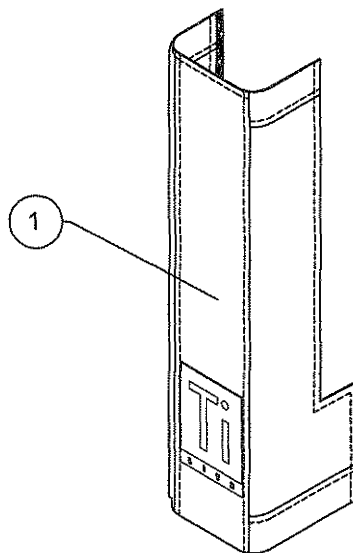

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103486	NEOPRENE SWINGAWAY IMPACT GUARD, RIGHT (10.50")
1B	1	CP103487	NEOPRENE SWINGAWAY IMPACT GUARD, LEFT (10.50")

REAR NEOPRENE IMPACT GUARDS

TiLite™ BB

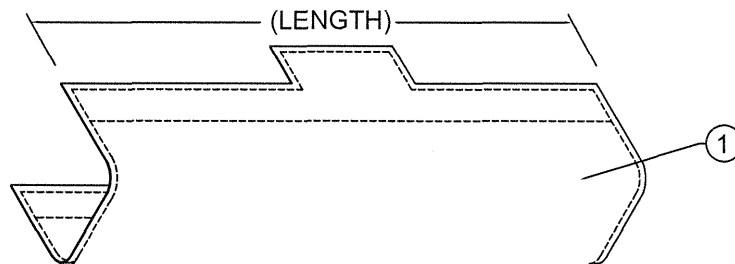


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	REAR FRAME PROTECTOR

TABLE ONE

SEAT WIDTH	LENGTH	PART NO.
12"	13.00"	CP100964
13"		
14"	15.00"	CP100966
15"		
16"	17.00"	CP100968
17"		
18"	19.00"	CP100969
19"		
20"	20.50"	CP101057

SEATBELTS


CHAIRS WITH BOLT ON SEAT SLING

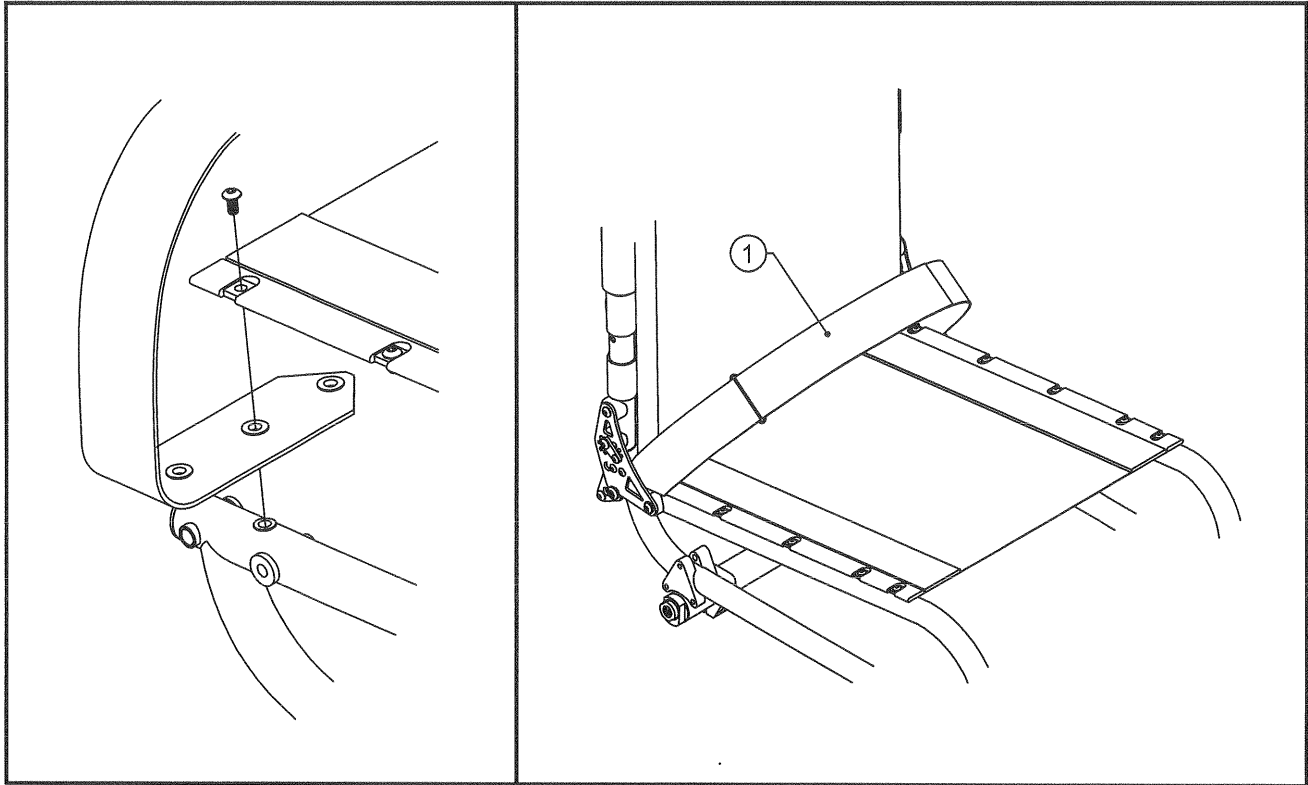

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP100501	VELCRO SEAT BELT

SEATBELTS


CHAIRS WITH TENSION ADJ BY STRAP SEAT SLING AND FOLDING ANG. ADJ. BACKS

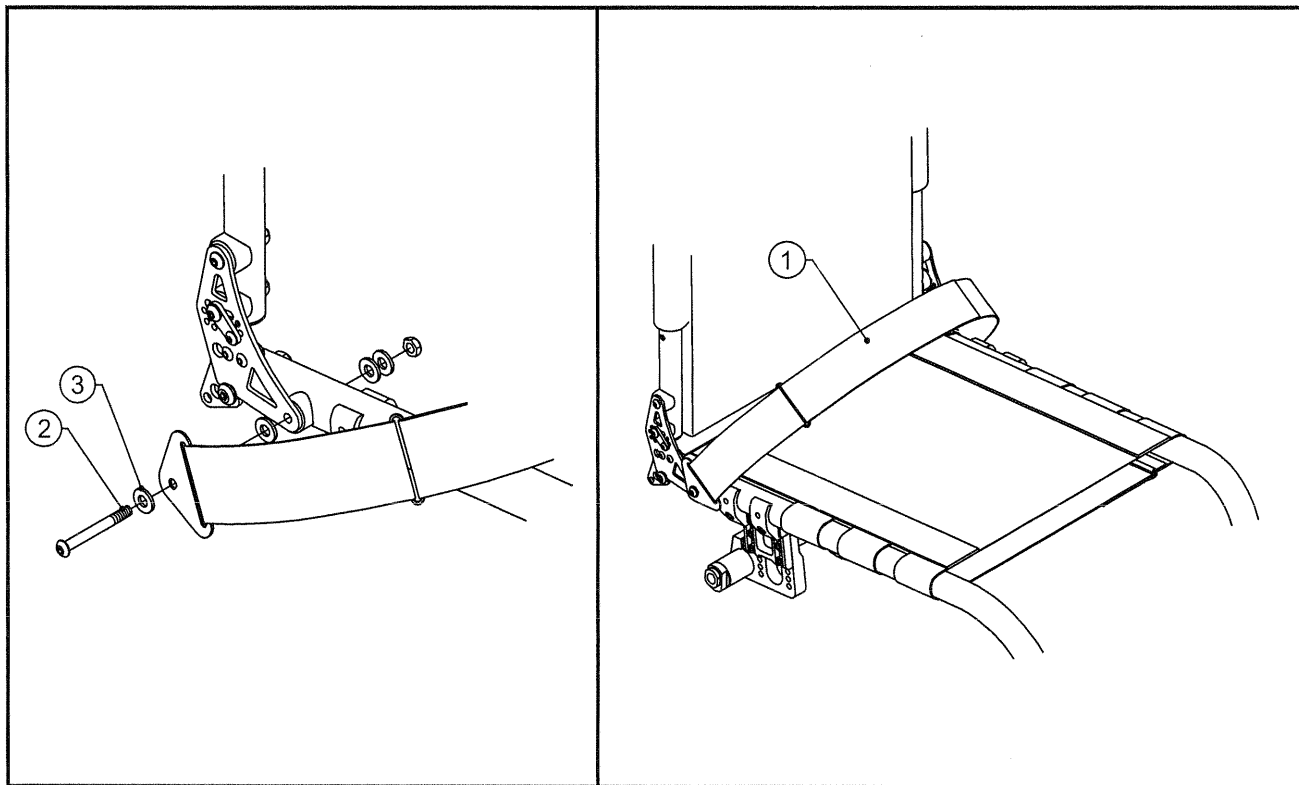

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP104222	VELCRO SEATBELT WITH BODYPOINT BRACKET
2	2	HD100105	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BUTTON HEAD CAP SCREW
3	2	HD100067	NYLON WASHER .500 OD X .265 ID X .031
2-3		AS104934	HARDWARE PACKAGE FOR ADDING ON SEATBELTS

SEATBELTS


TiLite Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

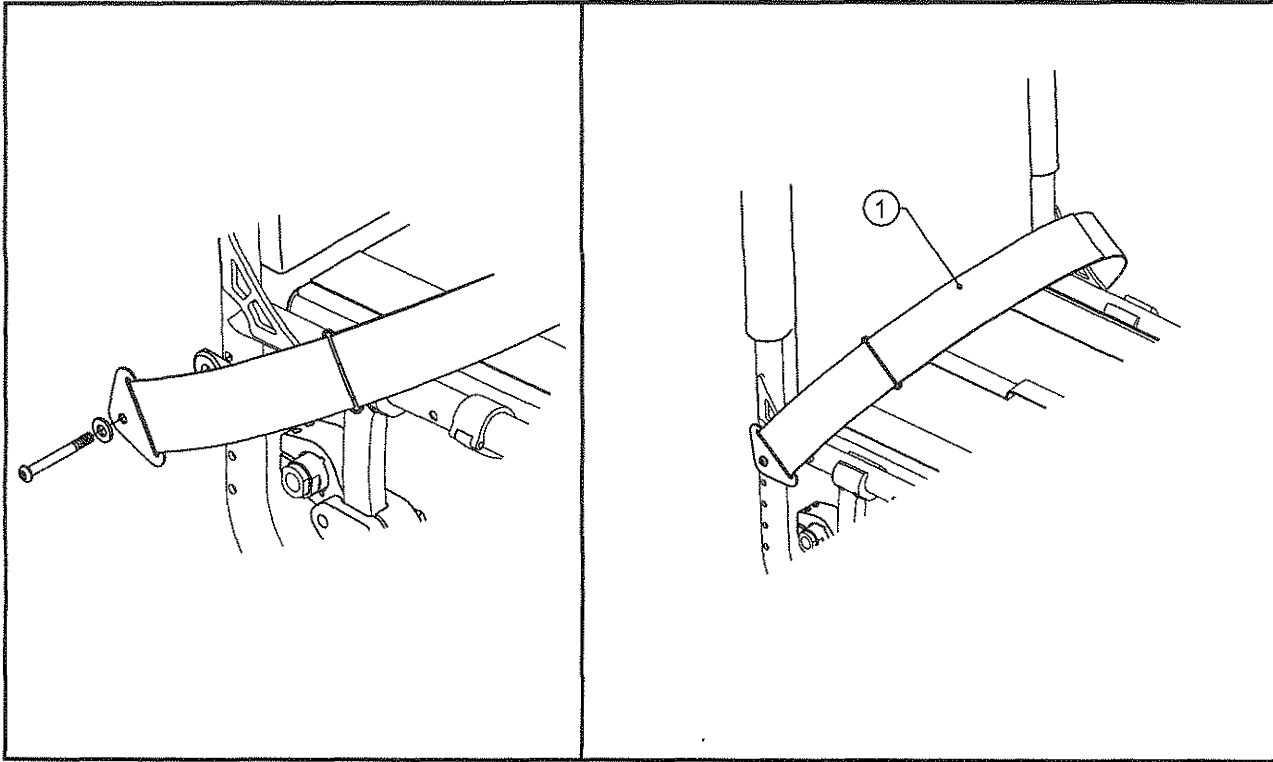

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP104222	VELCRO SEATBELT WITH BODYPOINT BRACKET

SEATBELTS


TiLite TX, ALSO FIXED BACK TR & YR W/ TENSION ADJ BY STRAP SEAT SLING

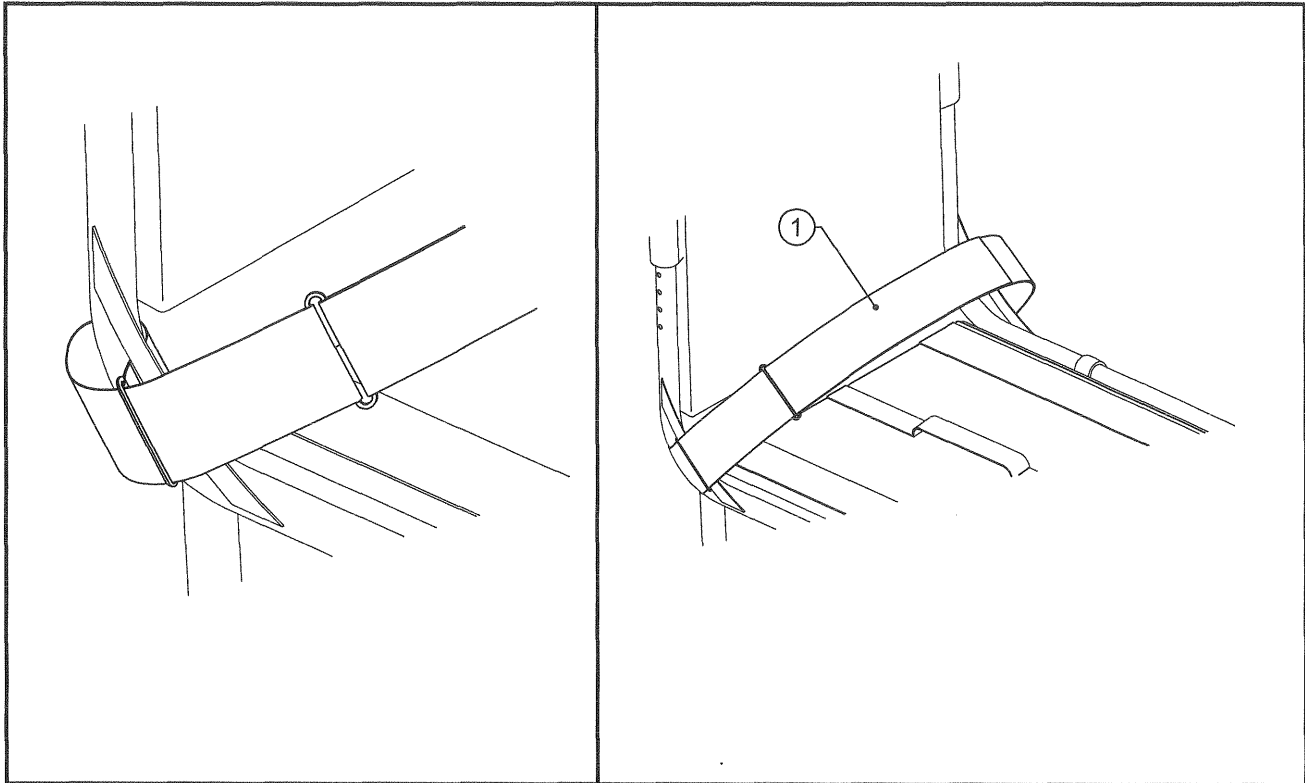

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP100905	VELCRO SEATBELT W/ METAL LOOPS

SEATBELTS


ZR (SERIES 2), ZRA(SERIES 2), AERO Z(SERIES 2), TR(SERIES 3), AERO T, TRA(SERIES 2)
WITH BY STRAP SEAT SLING AND FOLDING ANG. ADJ. BACKS, AND TWIST

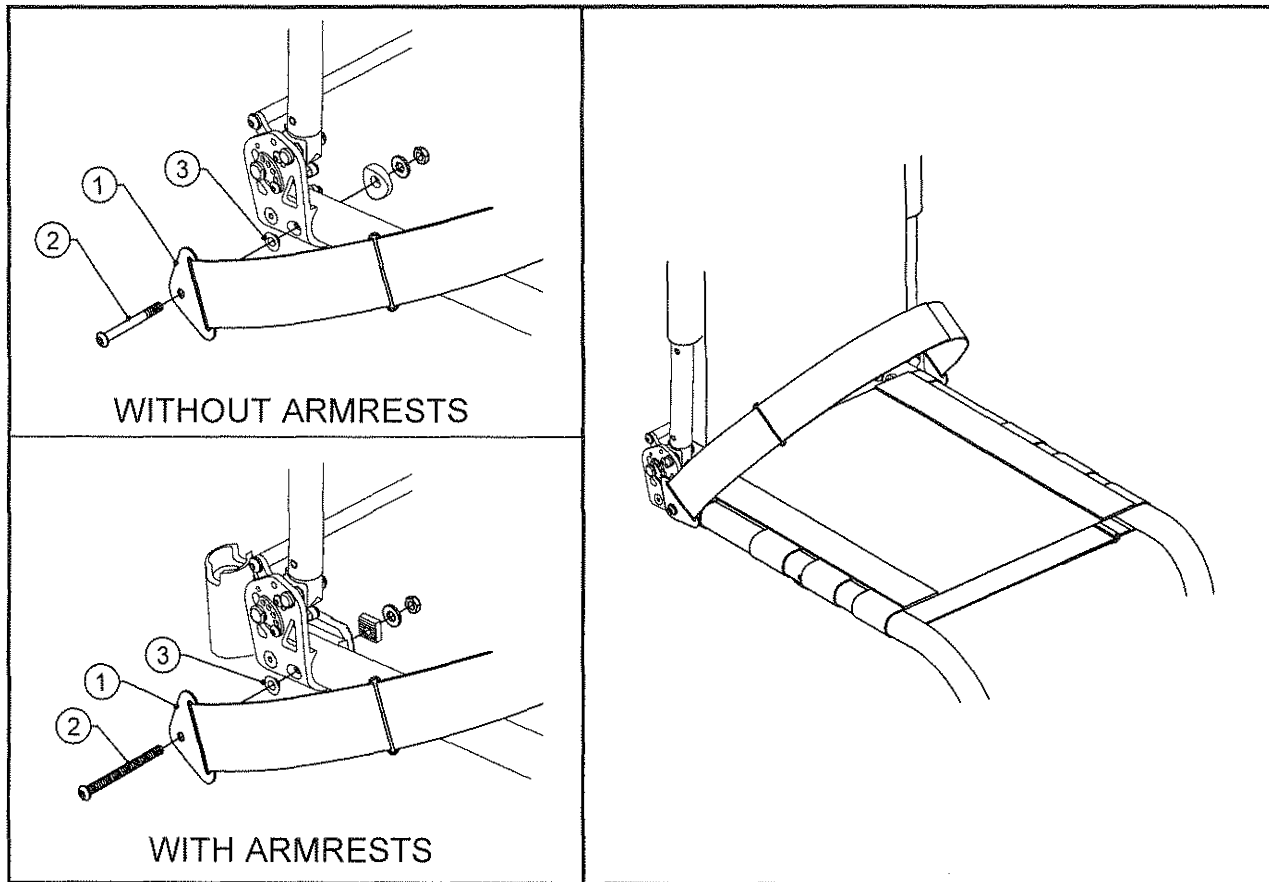

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP104222	VELCRO SEATBELT WITH BODYPOINT BRACKET
1B	1	CP100852	AUTO STYLE RELEASE BUCKLE SEAT BELT
1C	1	CP105851	BODYPOINT HIP BELT, PUSH-BUTTON, SMALL, 8"-12" WIDTHS
1D	1	CP105852	BODYPOINT HIP BELT, PUSH-BUTTON, MEDIUM, 13"-15" WIDTHS
1E	1	CP105853	BODYPOINT HIP BELT, SIDE RELEASE, SMALL, 8"-12" WIDTHS
1F	1	CP105854	BODYPOINT HIP BELT, SIDE RELEASE, MEDIUM, 13"-15" WIDTHS
1G	1	CP105855	BODYPOINT HIP BELT, REHAB LATCH, SMALL, 8"-12" WIDTHS
1H	1	CP105856	BODYPOINT HIP BELT, REHAB LATCH, MEDIUM, 13"-15" WIDTHS
2A	2	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
2B	2	HD100491	1/4-28 X 2-3/4" BUTTON HEAD CAP SCREW, BLACK ZINC
2C	2	HD100533	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, 1/2" THREAD, BLACK ZINC
3	2	CP104721	1/4" FLATHEAD SPACER
2A,3		AS110971	HARDWARE PACKAGE FOR ADDING ON SEATBELTS ON 1.25" CHAIRS WITHOUT ARMRESTS, AND 1" CHAIRS WITH ARMRESTS
2B,3		AS111171	HARDWARE PACKAGE FOR ADDING ON SEATBELTS WITH ARMRESTS ON 1.25" CHAIRS
2C,3		AS111995	HARDWARE PACKAGE FOR ADDING ON SEATBELTS ON 1" CHAIRS

SEATBELTS BODYPOINT EVOFLEX


ZR (SERIES 2), ZRA(SERIES 2), AERO Z(SERIES 2), TR(SERIES 3), AERO T, TRA(SERIES 2)
TWIST, 2GX, AERO X, AERO R, TX

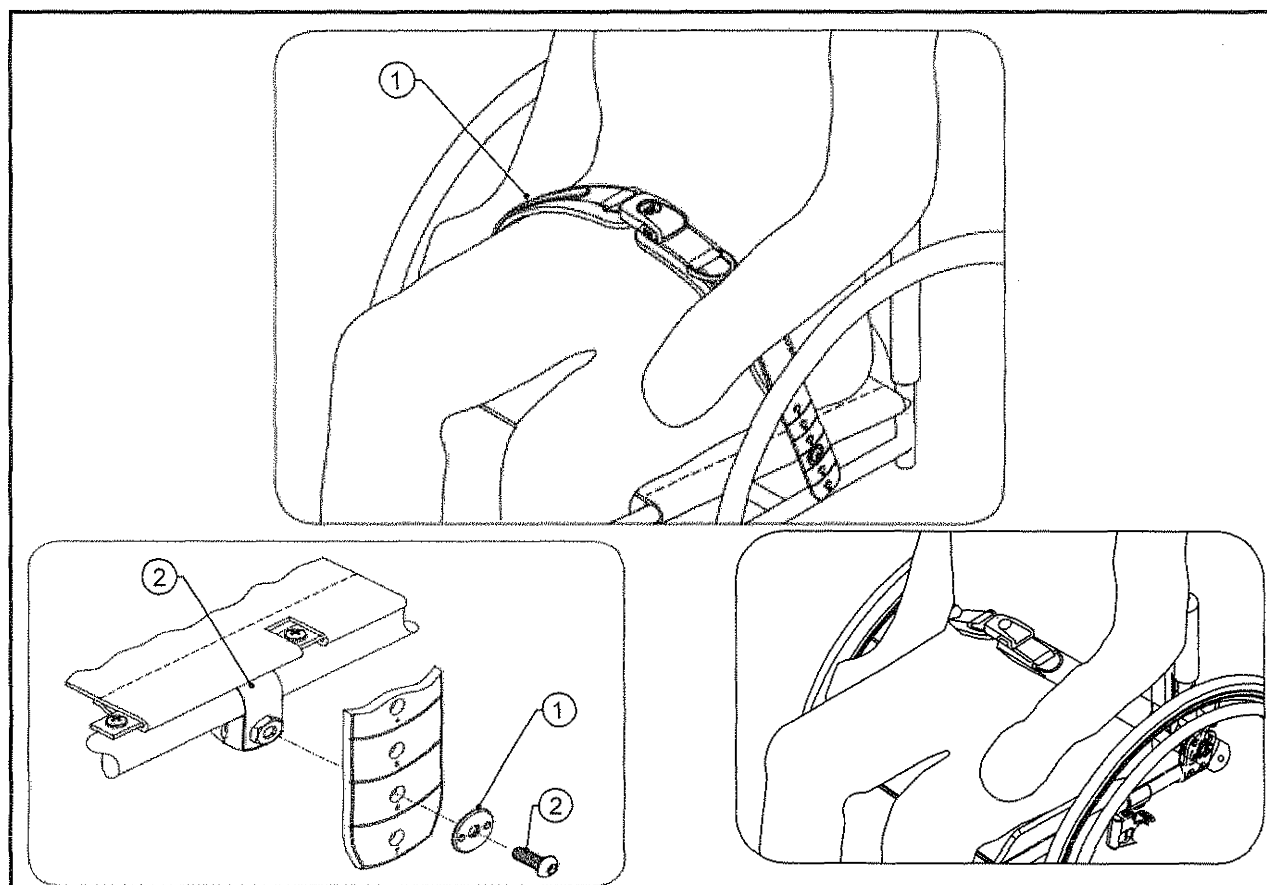

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP107399	EB235XS-20 EVOFLEX PELVIC STABILIZER X-SMALL (8"-12" SEAT WIDTHS)
1B	1	CP107400	EB205M-20 EVOFLEX PELVIC STABILIZER MEDIUM (13"-17" SEAT WIDTHS)
1C	1	CP107401	EB205L-20 EVOFLEX PELVIC STABILIZER LARGE (18"-22" SEAT WIDTHS)
2A	1	CP107402	HARDWARE PACKAGE HW320-25-10 BAND CLAMP 25 MM (1.0 in) DIA EVOFLEX
2B	1	CP107403	HARDWARE PACKAGE HW320-32-10 BAND CLAMP 32 MM (1.25 in) DIA EVOFLEX

CALF STRAP


ALL TiLite™ CHAIRS

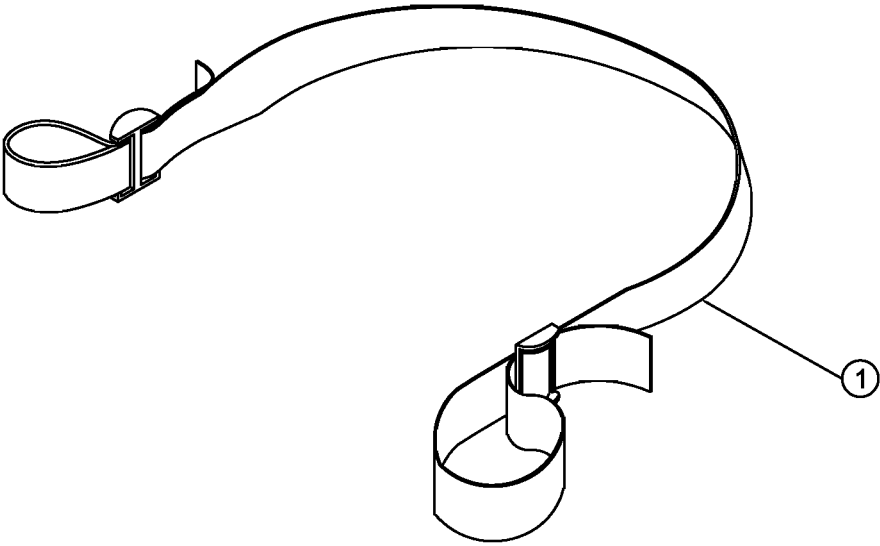

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100051	BUCKLE ADJUSTABLE CALFSTRAP

VELCRO CALF STRAP

ALL TiLite™ CHAIRS

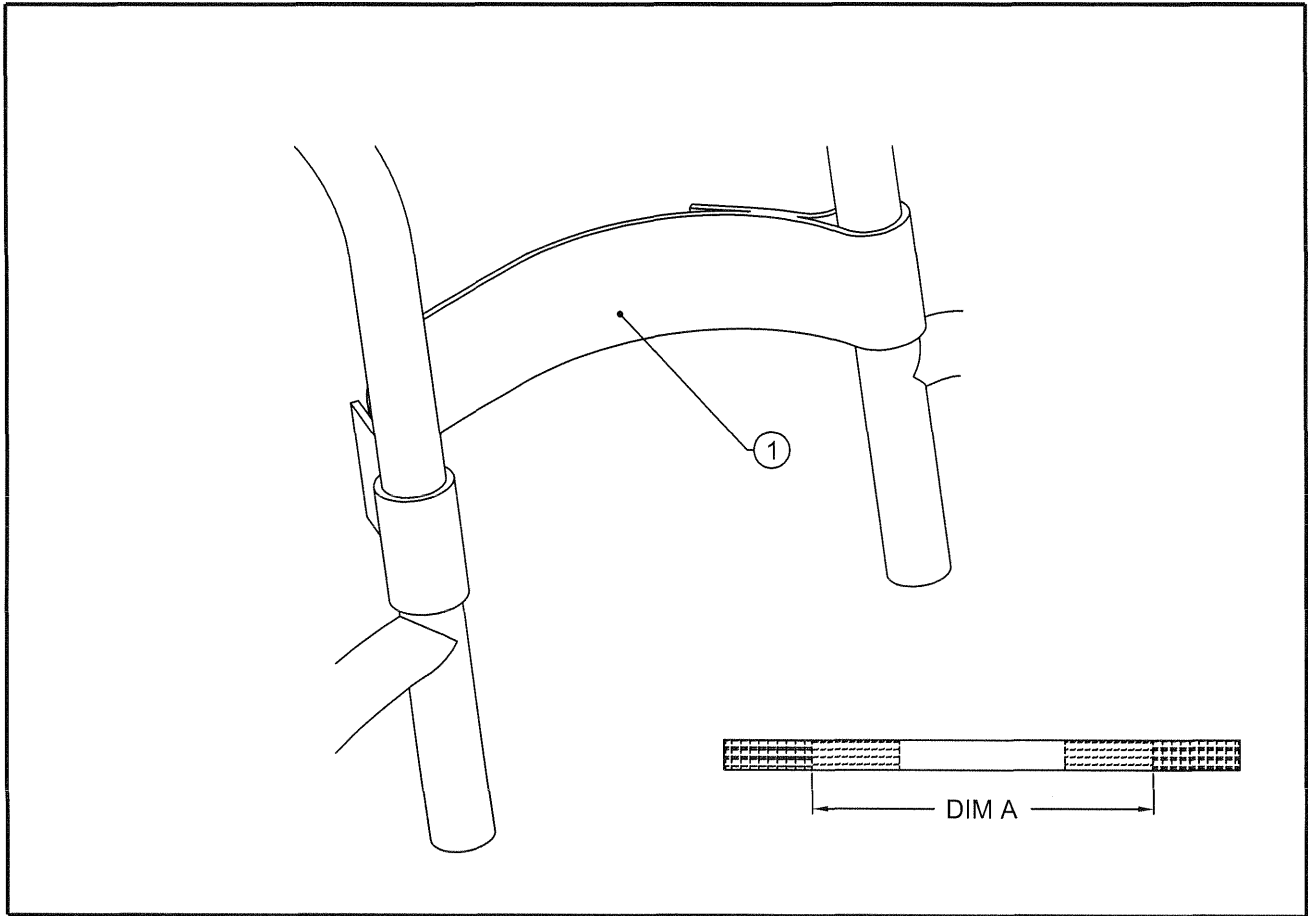


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	SEE TABLE	VELCRO CALF STRAP

PART NO.	DIM A	SUGGESTED FOOTREST WIDTHS
CP102418	13"	6"-9"
CP102516	16"	9"-12"
CP102517	20"	11"-15"
CP102518	23"	14"-17"
CP102519	26"	16"-19"

PADDED VELCRO CALF STRAP

ALL TiLite™ CHAIRS

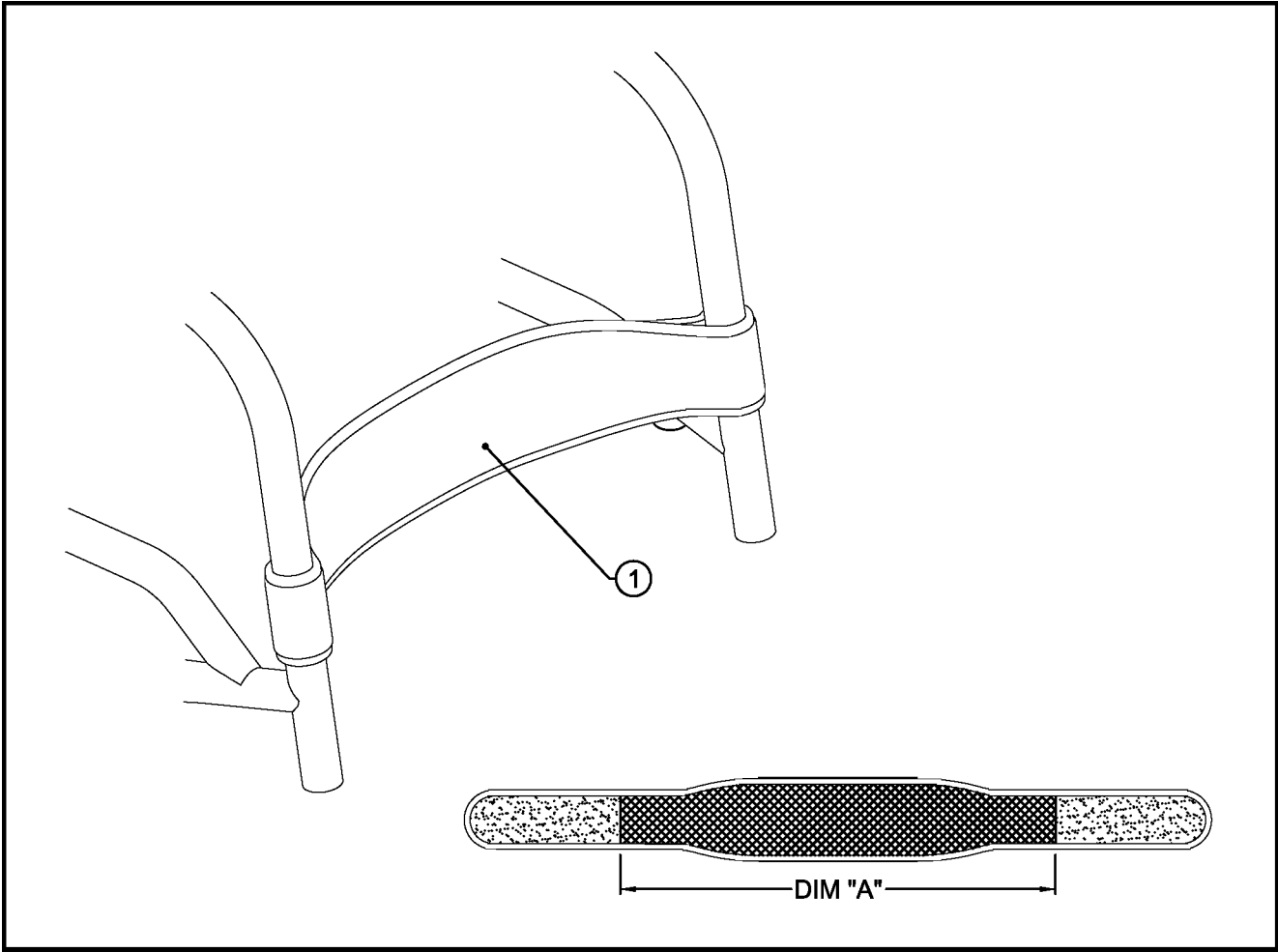


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	SEE TABLE	PADDED VELCRO CALF STRAP

TABLE ONE - PADDED VELCRO CALF STRAP

PART NO.	DIM "A"	SUGGESTED FOOTREST WIDTHS
CP103176	19"	8" - 11"
CP103177	23"	12" - 15"
CP103178	27"	16" - 20"

BACK PACKS AND SEAT POUCHES

ALL TiLite™ CHAIRS

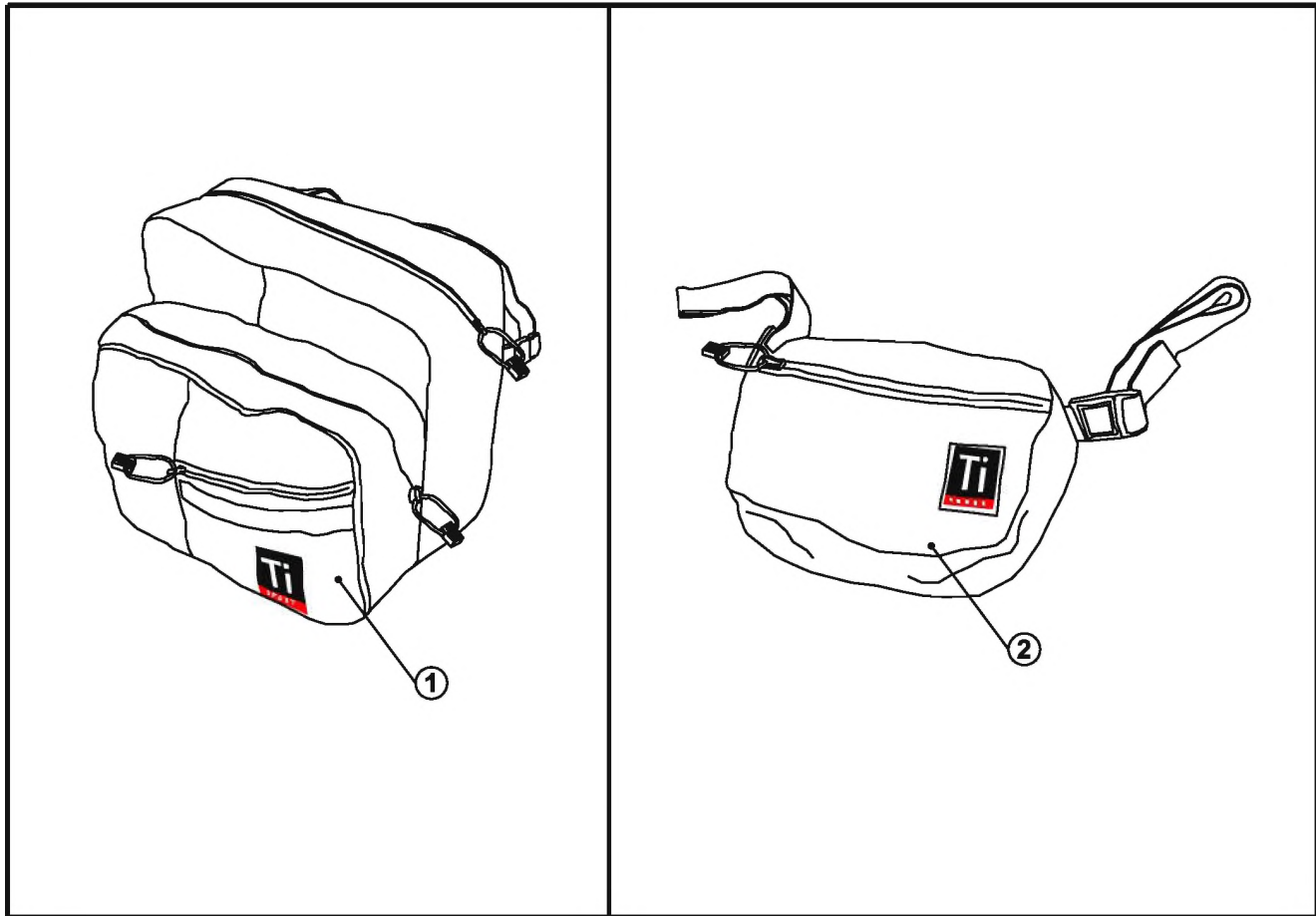


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP101036	BACK PACK SMALL (10-3/4" WIDE X 5-1/2" DEEP X 12-1/2" TALL)
1B	1	CP100261	BACK PACK LARGE (13" WIDE X 8" DEEP X 12-1/2" TALL)
2A	1	CP100260	SEAT POUCH SMALL (8-1/2" WIDE X 2" DEEP X 6-1/2" TALL)
2B	1	CP100929	SEAT POUCH LARGE (11-1/4" WIDE X 3-1/2" DEEP X 7" TALL)

PLEASE NOTE THAT ALL DIMENSIONS LISTED ARE APPROXIMATE

SPOKE GUARDS


ALL TiLite™ CHAIRS

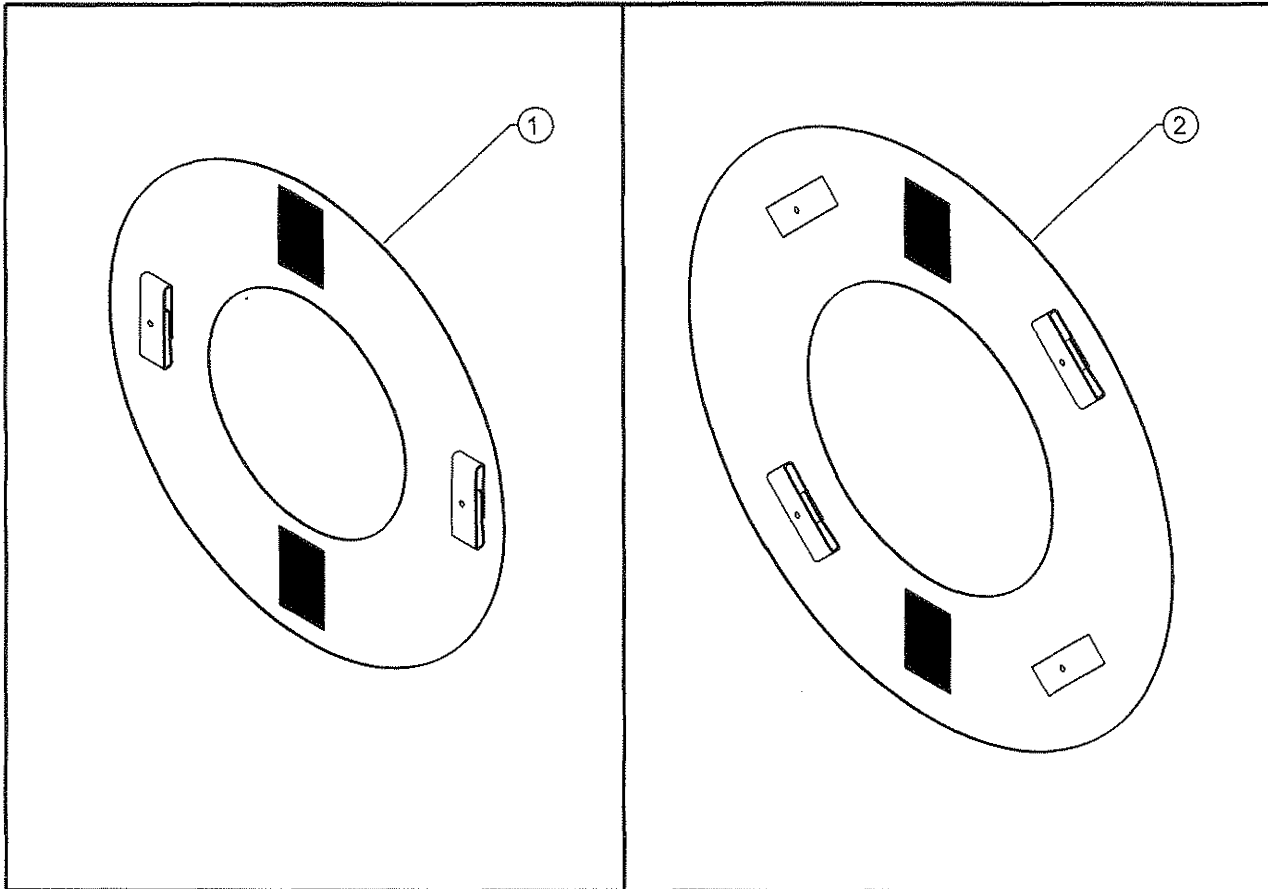

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A		AS117081	SPOKE GUARD 20" ASSEMBLY, PAIR
1B		AS102861	SPOKE GUARD 22" ASSEMBLY, PAIR
2A		AS102862	SPOKE GUARD 24/25" ASSEMBLY, PAIR
2B		AS102863	SPOKE GUARD 26" ASSEMBLY, PAIR

SPOKE GUARDS

ALL TiLite™ CHAIRS

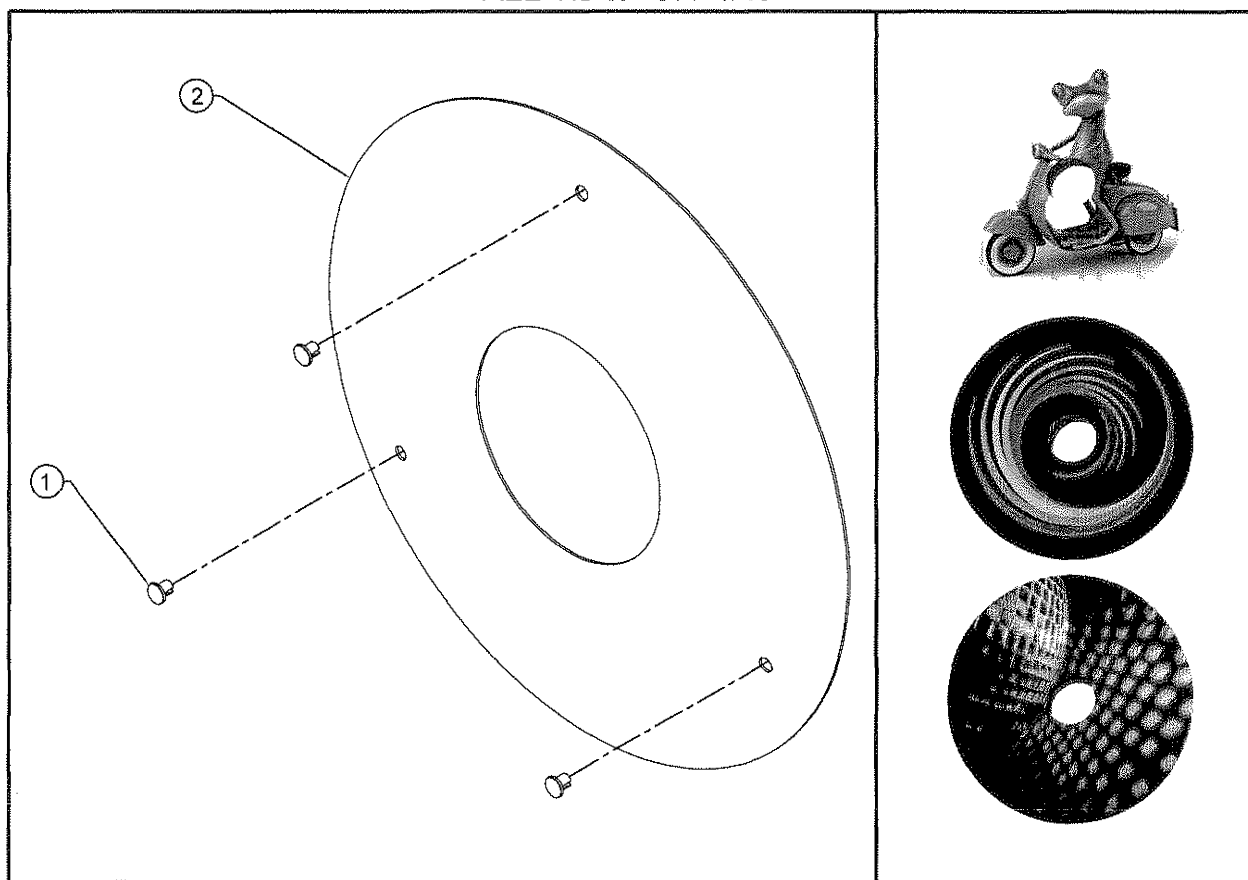


ITEM	QUANTITY	PART NO.	DESCRIPTION
1		CP107138	SPOKE GUARD SNAP CLAMP
2		TABLE ONE	SEE BELOW

TABLE ONE - SPOKEGAURDS

WHEEL SIZE	SCOOTER FROG	COLOR SPECTRUM	PURPLE PARTY
20"	CP107129	CP107132	CP107135
22"	CP107130	CP107133	CP107136
24"	CP107131	CP107134	CP107137

FRONT ANTI-TIP

TiLite™ TR, TRA, EVO, YR, YRA, ZR, ZRA, TR(SERIES2), YR(SERIES2),
AERO Z, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2),
TR (SERIES 3), AERO T, TRA (SERIES 2)

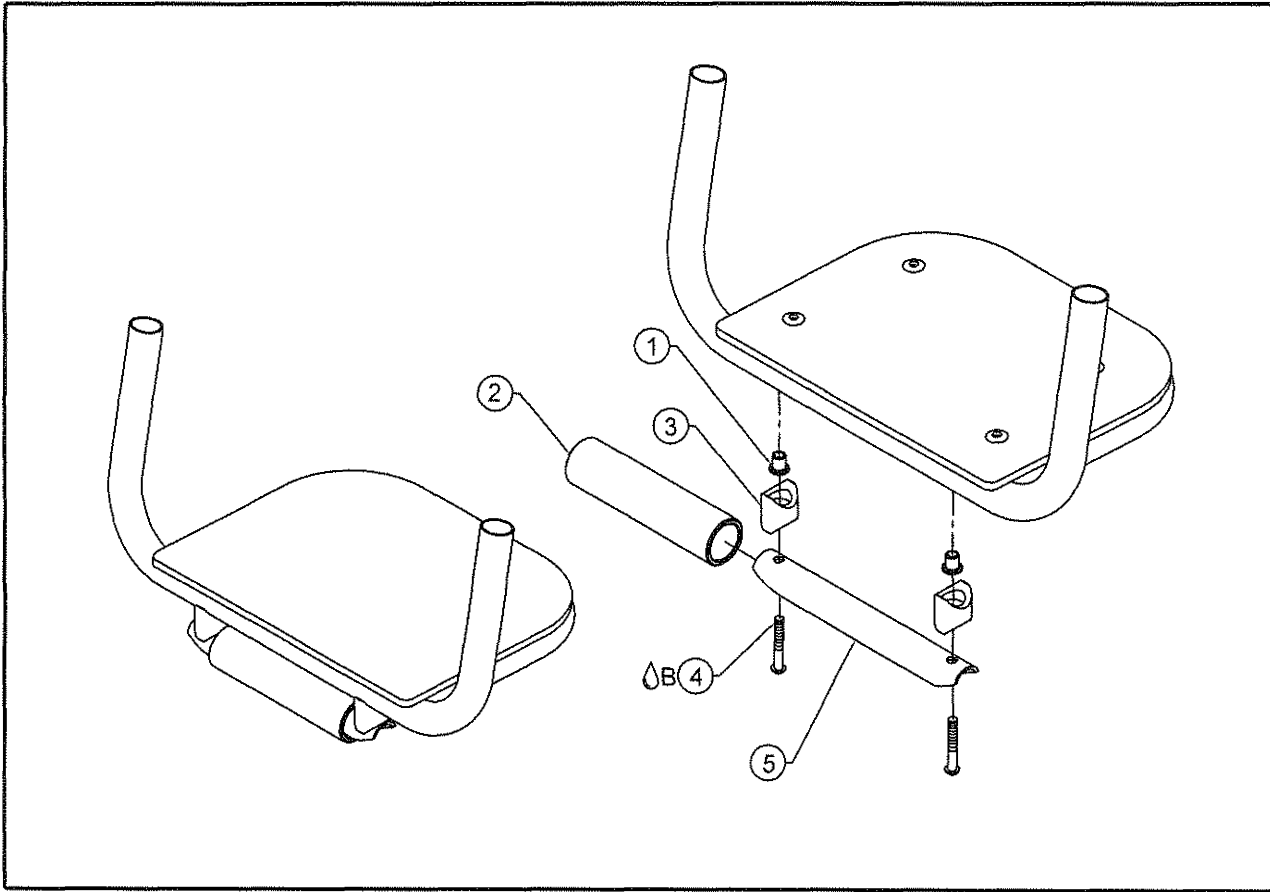


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	HD100154	RIVNUT
2	1	CP100847	ROLLER
3	2	CP100909	ROLLER STANDOFF 3/4 TO 3/4
4 	2	HD100224	10-32 X 1" BUTTON HEAD CAP SCREW
5	1	CP100848	ROLLER MOUNT
1-5		AS100563	FRONT ANTI-TIP

* FRONT ANTI-TIP MUST BE FACTORY INSTALLED

BASKETBALL ROLLER


TiLite™ BB

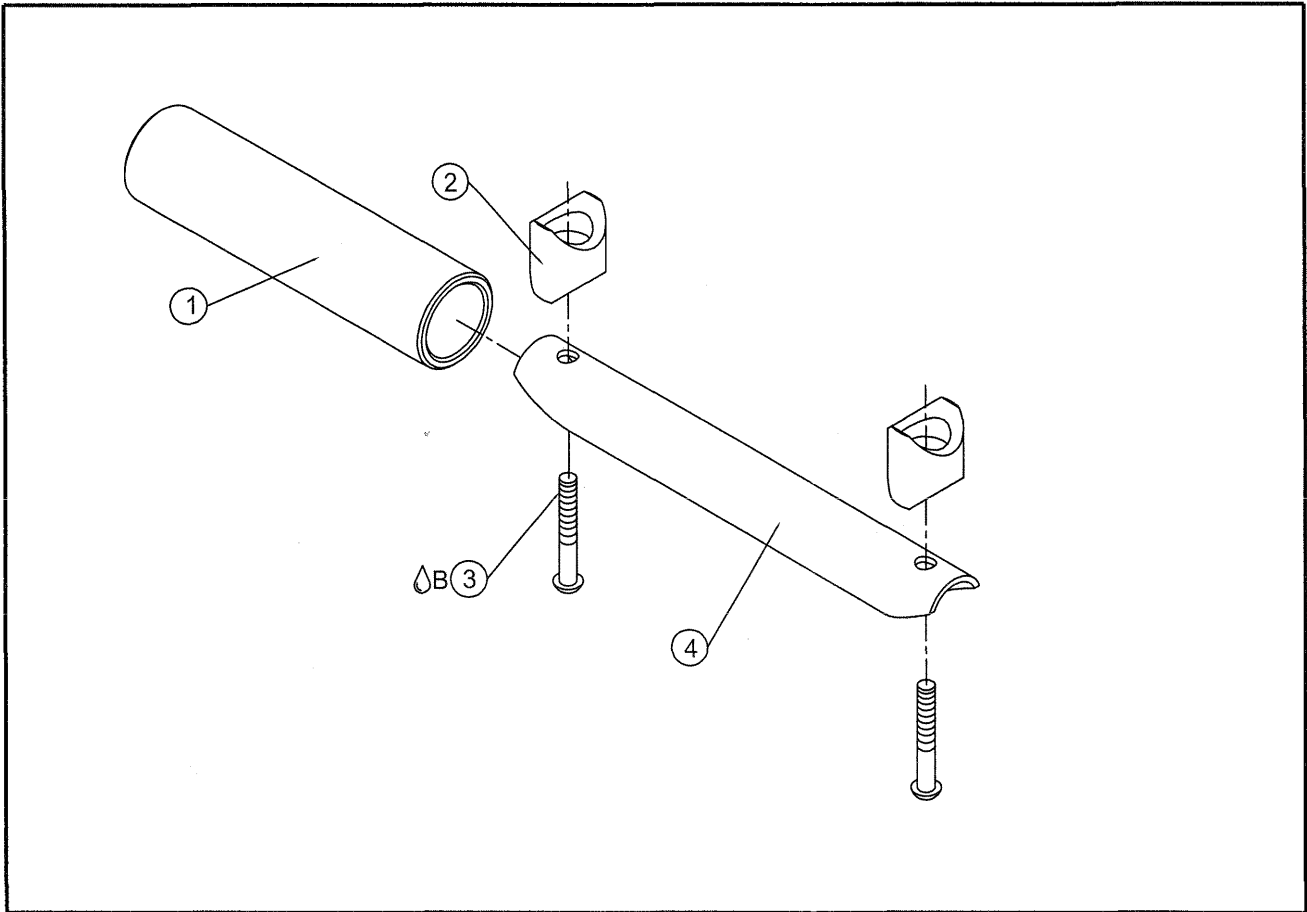

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100847	ROLLER
2	2	CP100927	ROLLER STANDOFF 7/8 TO 3/4
3 	2	HD100224	10-32 X 1" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
4	1	CP100848	ROLLER MOUNT
1-4		AS100564	BASKETBALL ROLLER ASSY 7/8-3/4

FRONT ROLLERS ON ANGLE ADJ FOOTRESTS

TiLite® BB, TR, ZRA, ZR, Aero Z, YR, ZR (SERIES 2),
ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2)

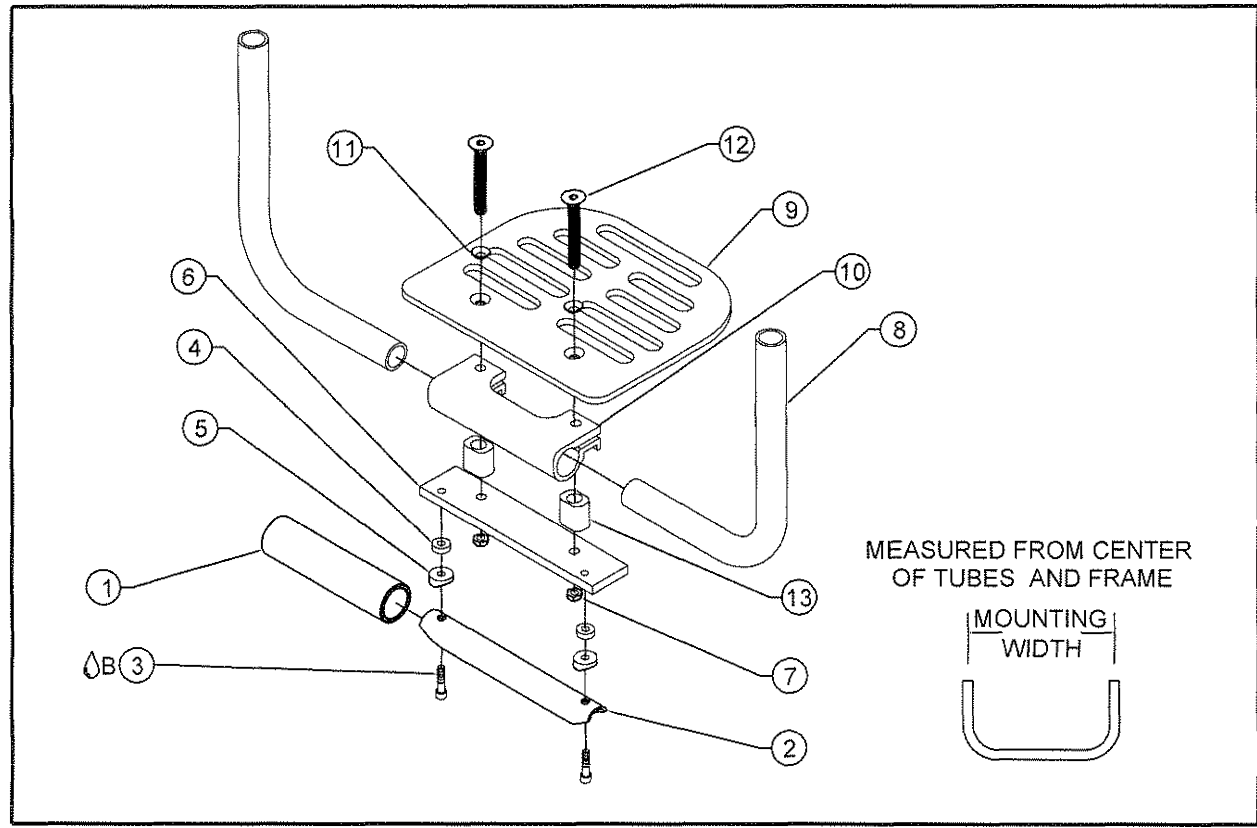

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100847	FRONT ROLLER
2	1	CP100848	FRONT ROLLER MOUNT
3	2	HD100341	1/4-20 X 3/5" SOCKET HEAD CAP SCREW, BLACK ZINC PLATE
4	2	CP102035	NYLON SPACER .25ID X .75OD X .25TH
5	2	HD100119	COVED WASHER 3/4" OD X 1/4" ID X 1/8" THK
6	1	CP104131	FRONT ROLLER PLATE ON ANGLE ADJUSTABLE FOOTREST
7	2	HD100333	1/4-28 X .218 NYLOCK LOCKNUT, BLACK ZINC PLATED
8	2	SEE BELOW	ANGLE ADJ FOOTREST EXTENSION (TABLE ONE)
9	1	SEE BELOW	ANGLE ADJ FOOTPLATE MOUNT (TABLE TWO)
10	1	CP107230	ANGLE ADJUSTABLE FOOTREST CLAMP LITE
11	2	HD100526	1/4" COUNTERSUNK WASHER
12	2	HD100651	1/4-28 X 1-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
13	2	CP107420	FRONT ROLLER SPACER
1-13		SEE BELOW	FRONT ROLLERS ON ANGLE ADJ FOOTRESTS ASSY (TABLE ONE)

TABLE ONE

FOOTPLATE ASSY. PART#	FOOTPLATE EXTENSION PART#	1" DIAMETER TUBE CHAIR MOUNTING WIDTH	1.25" DIAMETER TUBE CHAIR MOUNTING WIDTH
AS104827	CP100859	10"-11"	10 1/4"-11 1/4"
AS104828	CP100859	12"	11 1/4"-12 1/4"
AS104829	CP100860	13"	13 1/4"-14 1/4"
AS104830	CP100860	14"-15"	14 1/4"-15 1/4"
AS104831	CP100861	16"-18"	16 1/4"-17 1/4"
AS104832	CP101948	19"-20"	N/A


TABLE TWO

FOOTPLATE PART#	1" DIAMETER TUBE CHAIR MOUNTING WIDTH	1.25" DIAMETER TUBE CHAIR MOUNTING WIDTH
CP100855	10"-11"	10 1/4"-11 1/4"
CP100856	12"-13"	11 1/4"-12 1/4"
CP100857	14"-15"	13 1/4"-14 1/4"
CP100858	16"-17"	14 1/4"-15 1/4"
CP101983	18"-19"	16 1/4"-17 1/4"



FRONT ROLLERS ON ANGLE ADJ FOOTRESTS

TiLite™ Aero R

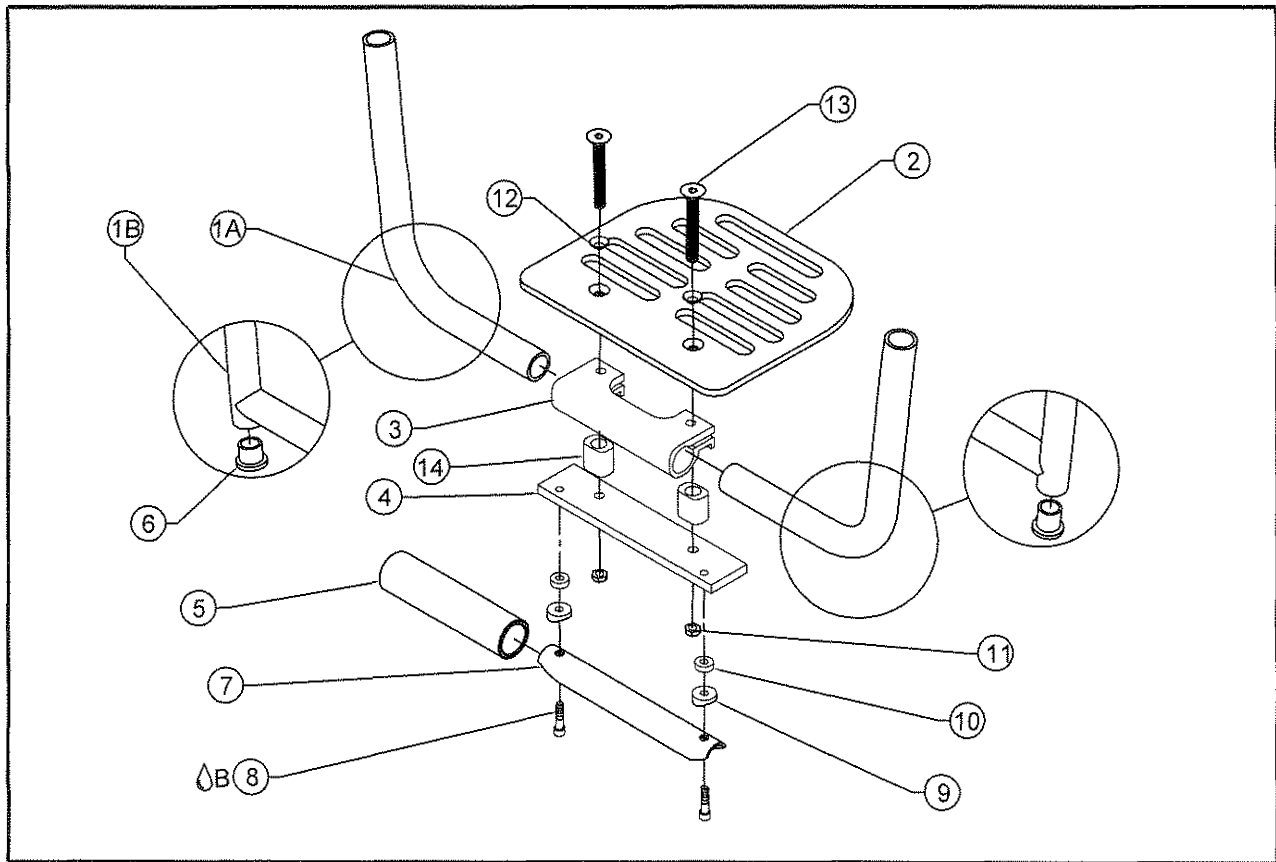

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	2	SEE BELOW	ANGLE ADJ FOOTPLATE EXTENSION
1B	2	SEE BELOW	ANGLE ADJ FOOTPLATE EXTENSION
2	1	SEE BELOW	ANGLE ADJ FOOTPLATE
3	1	CP107230	ANGLE ADJUSTABLE FOOTREST CLAMP LITE
4	1	CP104131	FRONT ROLLER PLATE ON ANGLE ADJUSTABLE FOOTREST
5	1	CP100847	FRONT ROLLER
6	2	HD100127	END CAP 3/4" X .083 WALL TUBING
7	1	CP100848	FRONT ROLLER MOUNT
8A	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
9	2	HD100119	COVED WASHER 3/4" OD X 1/4" ID X 1/8" THK
10	2	CP102035	NYLON SPACER .25ID X .75OD X .25TH
11	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
12	2	HD100526	1/4" COUNTERSUNK WASHER
13	2	HD100651	1/4-28 X 1-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
14	2	CP107420	FRONT ROLLER SPACER
1-14		SEE BELOW	FRONT ROLLERS ON ANGLE ADJ FOOTRESTS ASSY


FOOTPLATE ASSY#	BENT/ FOOTPLATE EXTENSION PART#	MITERED/ FOOTPLATE EXTENSION PART#	FOOTPLATE PART#	CHAIR WIDTH
AS104866	N/A	CP102025	CP100855	14"-15"
AS104867	CP102022	N/A	CP100855	16"
AS104868	CP102022	N/A	CP100856	17"
AS104869	CP102023	N/A	CP100857	18"
AS104870	CP102023	N/A	CP100858	19"
AS104871	CP102024	N/A	CP101983	20"



Parts Manual

HIGH MNT ANG ADJ. FOOTPLATE W/ FRONT ROLLRS

TiLite™ ZR, ZRA, AERO Z, Z10, TR(SERIES 2), YR (SERIES 2),
ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2)

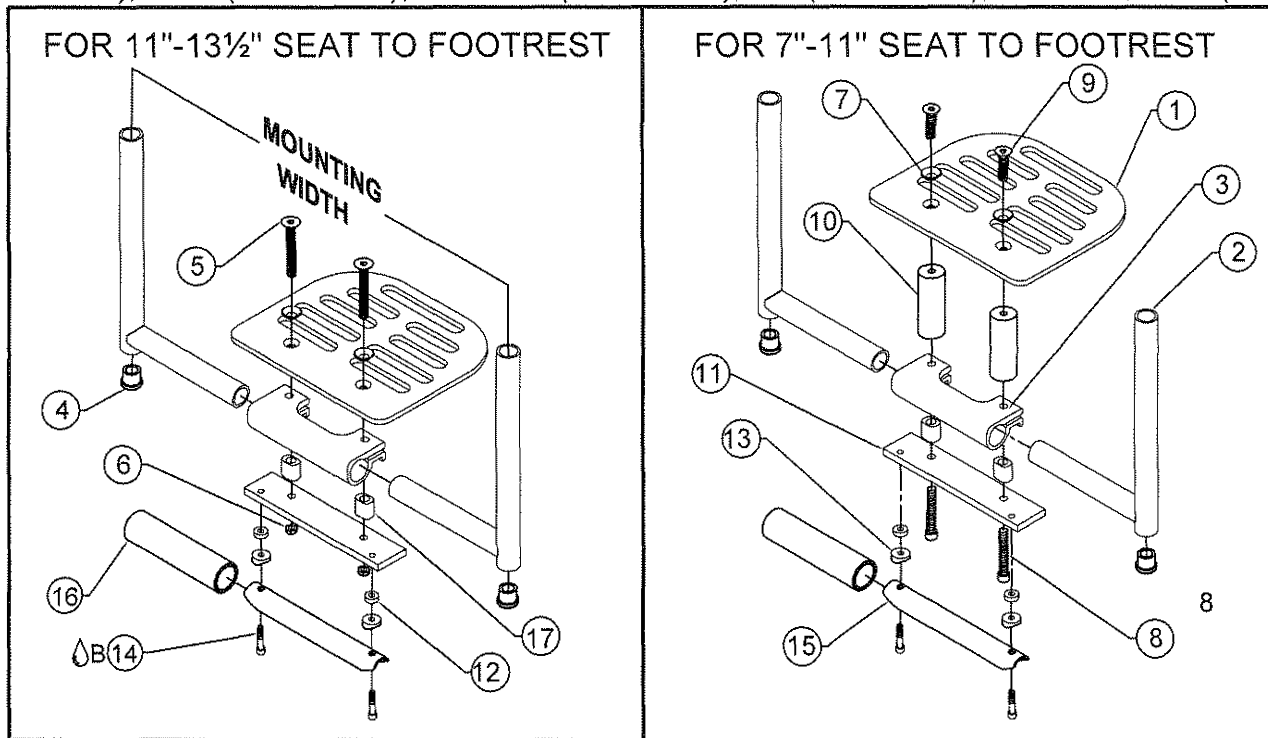

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP100855	ANGLE ADJUSTABLE FOOT PLATE 10¼"-11¼" MOUNTING WIDTH
1B	1	CP100856	ANGLE ADJUSTABLE FOOT PLATE 11¼"-12¼" MOUNTING WIDTH
1C	1	CP100857	ANGLE ADJUSTABLE FOOT PLATE 12¼"-13¼" MOUNTING WIDTH
1D	1	CP100858	ANGLE ADJUSTABLE FOOT PLATE 13¼"-14¼" MOUNTING WIDTH
1E	1	CP101983	ANGLE ADJUSTABLE FOOT PLATE 14¼"-16¼" MOUNTING WIDTH
2A	2	CP100862	ANGLE ADJ FOOTREST EXT HIGH MNT 10-12 MNT WD
2B	2	CP100863	ANGLE ADJ FOOTREST EXT HIGH MNT 13-15 MNT WD
2C	2	CP100864	ANGLE ADJ FOOTREST EXT HIGH MNT 16-18 MNT WD
3	1	CP107230	ANGLE ADJUSTABLE FOOTREST CLAMP LITE
4	2	HD100127	END CAP 3/4" X .083 WALL TUBING
5	2	HD100651	1/4-28 X 1-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	2	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
7	2	HD100526	1/4" COUNTERSUNK WASHER
8	2	HD100068	1/4-20 X 1-1/4" LONG SOCKET HEAD CAP SCREW, CLEAR ZINC
9	2	HD100336	1/4-20 X 3/4" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
10A	2	CP102078	2" HIGHMOUNT FOOTPLATE EXTENSION
10B	2	CP102079	4" HIGHMOUNT FOOTPLATE EXTENSION
11	1	CP104131	FRONT ROLLER PLATE ON ANGLE ADJUSTABLE FOOTREST
12	2	CP102035	NYLON SPACER .25ID X .75OD X .25TH
13	2	HD100119	COVED WASHER 3/4" OD X 1/4" ID X 1/8" THK
14B	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
15	1	CP100848	FRONT ROLLER MOUNT
16	1	CP100847	FRONT ROLLER
17	2	CP107420	FRONT ROLLER SPACER

1.25" DIAMETER TUBE CHAIR MOUNTING WIDTH	1" DIAMETER TUBE CHAIR MOUNTING WIDTH
10½"-11¼"	10"-11"
11¼"-12¼"	11"-12"
12¼"-13¼"	12"-13"
13¼"-14¼"	13"-14"
14¼"-16¼"	14"-16"

HIGH MNT ANG ADJ. FOOTPLATE W/ FRONT ROLLERS (CONTINUED)

TiLite™ ZR, ZRA, AERO Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3-7,11-16		AS104901	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 10¼"-11¼" MNT WIDTH, 11"-13" SEAT TO FTRST
1B,2A,3-7,11-16		AS104902	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 11¼"-12¼" MNT WIDTH, 11"-13" SEAT TO FTRST
1C,2B,3-7,11-16		AS104903	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 13¼"-14¼" MNT WIDTH, 11"-13" SEAT TO FTRST
1D,2B,3-7,11-16		AS104904	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14¼"-15¼" MNT WIDTH, 11"-13" SEAT TO FTRST
1E,2C,3-7,11-16		AS104905	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16¼"-17¼" MNT WIDTH, 11"-13" SEAT TO FTRST
1A,2A,3-4,7-10A,11-16		AS104906	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 10¼"-11¼" MNT WIDTH, 9"-11" SEAT TO FTRST
1B,2A,3-4,7-10A,11-16		AS104907	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 11¼"-12¼" MNT WIDTH, 9"-11" SEAT TO FTRST
1C,2B,3-4,7-10A,11-16		AS104908	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 13¼"-14¼" MNT WIDTH, 9"-11" SEAT TO FTRST
1D,2B,3-4,7-10A,11-16		AS104909	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14¼"-15¼" MNT WIDTH, 9"-11" SEAT TO FTRST
1E,2C,3-4,7-10A,11-16		AS104910	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16¼"-17¼" MNT WIDTH, 9"-11" SEAT TO FTRST
1A,2A,3-4,7-10B,11-16		AS104911	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 10¼"-11¼" MNT WIDTH, 7"-9" SEAT TO FTRST
1B,2A,3-4,7-10B,11-16		AS104912	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 11¼"-12¼" MNT WIDTH, 7"-9" SEAT TO FTRST
1C,2B,3-4,7-10B,11-16		AS104913	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 13¼"-14¼" MNT WIDTH, 7"-9" SEAT TO FTRST
1D,2B,3-4,7-10B,11-16		AS104914	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14¼"-15¼" MNT WIDTH, 7"-9" SEAT TO FTRST
1E,2C,3-4,7-10B,11-16		AS104915	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16¼"-17¼" MNT WIDTH, 7"-9" SEAT TO FTRST


TiLite™ TR(SERIES 2), YR (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3-7,11-16		AS104901	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 10"-11" MNT WIDTH, 11"-13" SEAT TO FTRST
1B,2A,3-7,11-16		AS104902	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 11"-12" MNT WIDTH, 11"-13" SEAT TO FTRST
1C,2B,3-7,11-16		AS104903	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 13"-14" MNT WIDTH, 11"-13" SEAT TO FTRST
1D,2B,3-7,11-16		AS104904	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14"-15" MNT WIDTH, 11"-13" SEAT TO FTRST
1E,2C,3-7,11-16		AS104905	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16"-17" MNT WIDTH, 11"-13" SEAT TO FTRST
1A,2A,3-4,7-10A,11-16		AS104906	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 10"-11" MNT WIDTH, 9"-11" SEAT TO FTRST
1B,2A,3-4,7-10A,11-16		AS104907	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 11"-12" MNT WIDTH, 9"-11" SEAT TO FTRST
1C,2B,3-4,7-10A,11-16		AS104908	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 13"-14" MNT WIDTH, 9"-11" SEAT TO FTRST
1D,2B,3-4,7-10A,11-16		AS104909	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14"-15" MNT WIDTH, 9"-11" SEAT TO FTRST
1E,2C,3-4,7-10A,11-16		AS104910	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16"-17" MNT WIDTH, 9"-11" SEAT TO FTRST
1A,2A,3-4,7-10B,11-16		AS104911	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 10"-11" MNT WIDTH, 7"-9" SEAT TO FTRST
1B,2A,3-4,7-10B,11-16		AS104912	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 11"-12" MNT WIDTH, 7"-9" SEAT TO FTRST
1C,2B,3-4,7-10B,11-16		AS104913	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 13"-14" MNT WIDTH, 7"-9" SEAT TO FTRST
1D,2B,3-4,7-10B,11-16		AS104914	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14"-15" MNT WIDTH, 7"-9" SEAT TO FTRST
1E,2C,3-4,7-10B,11-16		AS104915	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16"-17" MNT WIDTH, 7"-9" SEAT TO FTRST



HIGH MNT ANG ADJ. FOOTPLATE W/ FRONT ROLLRS

TiLite™ Aero R

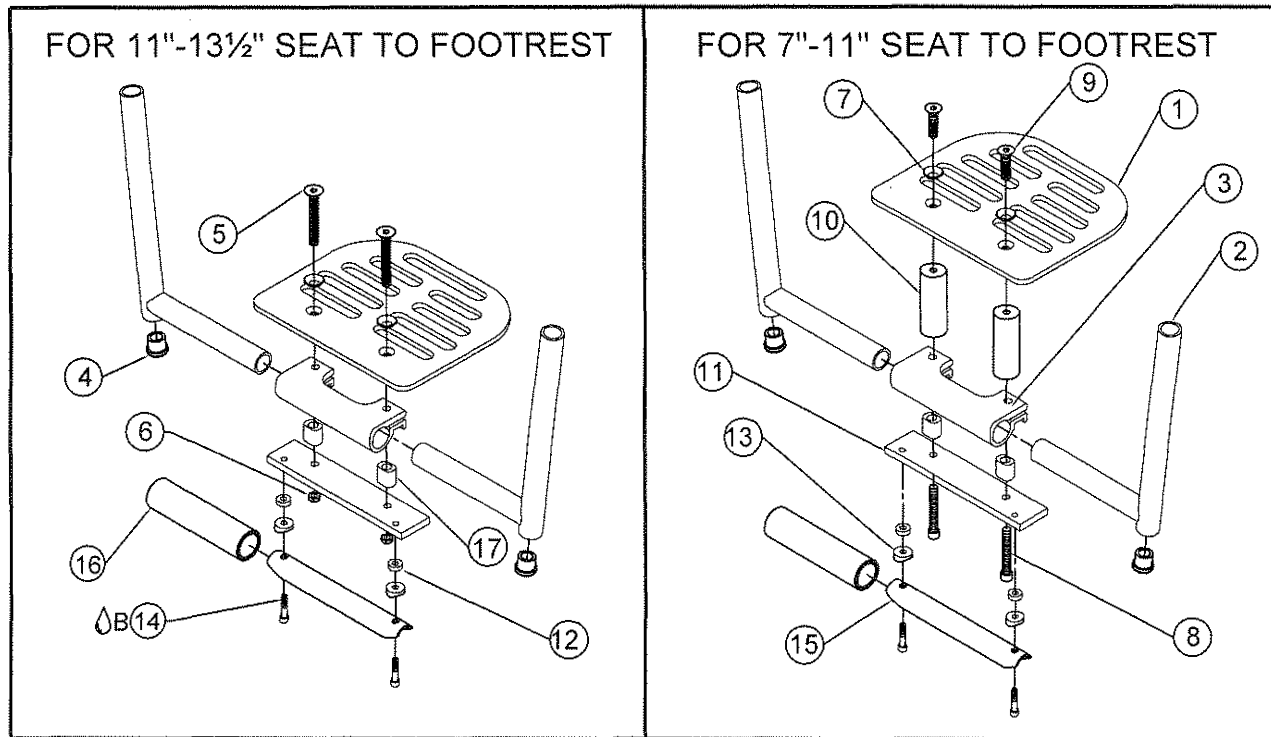

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP100855	ANGLE ADJUSTABLE FOOT PLATE 14" CHAIR WIDTH
1B	1	CP100856	ANGLE ADJUSTABLE FOOT PLATE 15" CHAIR WIDTH
1C	1	CP100857	ANGLE ADJUSTABLE FOOT PLATE 16" CHAIR WIDTH
1D	1	CP100858	ANGLE ADJUSTABLE FOOT PLATE 17"-20" CHAIR WIDTH
2A	2	CP102025	EVO HIGHMOUNT FOOTREST EXTENSION 14"-15" CHAIR WIDTH
2B	2	CP102026	EVO HIGHMOUNT FOOTREST EXTENSION 16"-17" CHAIR WIDTH
2C	2	CP102027	EVO HIGHMOUNT FOOTREST EXTENSION 18"-19" CHAIR WIDTH
2D	2	CP102028	EVO HIGHMOUNT FOOTREST EXTENSION 20" CHAIR WIDTH
3	1	CP107230	ANGLE ADJUSTABLE FOOTREST CLAMP LITE
4	2	HD100127	END CAP 3/4" X .083 WALL TUBING
5	2	HD100651	1/4-28 X 1-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	2	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
7	2	HD100526	1/4" COUNTERSUNK WASHER
8	2	HD100068	1/4-20 X 1-1/4" LONG SOCKET HEAD CAP SCREW, CLEAR ZINC
9	2	HD100336	1/4-20 X 3/4" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
10A	2	CP102078	2" HIGHMOUNT FOOTPLATE EXTENSION
10B	2	CP102079	4" HIGHMOUNT FOOTPLATE EXTENSION
11	1	CP104131	FRONT ROLLER PLATE ON ANGLE ADJUSTABLE FOOTREST
12	2	CP102035	NYLON SPACER .25ID X .75OD X .25TH
13	2	HD100119	COVED WASHER 3/4" OD X 1/4" ID X 1/8" THK
14 ^B	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
15	1	CP100848	FRONT ROLLER MOUNT
16	1	CP100847	FRONT ROLLER
17	2	CP107420	FRONT ROLLER SPACER

HIGH MNT ANG ADJ. FOOTPLATE W/ FRONT ROLLERS (CONTINUED)


TiLite™ AERO R

ITEM	QTY	PART NO.	DESCRIPTION
1A,2A,3-7,11-16		AS104916	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14" CHAIR WIDTH, 11"-13½" SEAT TO FTRST
1B,2A,3-7,11-16		AS104917	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 15" CHAIR WIDTH, 11"-13½" SEAT TO FTRST
1C,2B,3-7,11-16		AS104918	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16" CHAIR WIDTH, 11"-13½" SEAT TO FTRST
1D,2B,3-7,11-16		AS104919	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 17" CHAIR WIDTH, 11"-13½" SEAT TO FTRST
1D,2C,3-7,11-16		AS104920	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 18"-19" CHAIR WIDTH, 11"-13½" SEAT TO FTRST
1D,2D,3-7,11-16		AS104921	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 20" CHAIR WIDTH, 11"-13½" SEAT TO FTRST
1A,2A,3-4,7-10A,11-16		AS104922	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14" CHAIR WIDTH, 9"-11" SEAT TO FTRST
1B,2A,3-4,7-10A,11-16		AS104923	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 15" CHAIR WIDTH, 9"-11" SEAT TO FTRST
1C,2B,3-4,7-10A,11-16		AS104924	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16" CHAIR WIDTH, 9"-11" SEAT TO FTRST
1D,2B,3-4,7-10A,11-16		AS104925	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 17" CHAIR WIDTH, 9"-11" SEAT TO FTRST
1D,2C,3-4,7-10A,11-16		AS104926	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 18"-19" CHAIR WIDTH, 9"-11" SEAT TO FTRST
1D,2D,3-4,7-10A,11-16		AS104927	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 20" CHAIR WIDTH, 9"-11" SEAT TO FTRST
1A,2A,3-4,7-10B,11-16		AS104928	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 14" CHAIR WIDTH, 7"-9" SEAT TO FTRST
1B,2A,3-4,7-10B,11-16		AS104929	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 15" CHAIR WIDTH, 7"-9" SEAT TO FTRST
1C,2B,3-4,7-10B,11-16		AS104930	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 16" CHAIR WIDTH, 7"-9" SEAT TO FTRST
1D,2B,3-4,7-10B,11-16		AS104931	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 17" CHAIR WIDTH, 7"-9" SEAT TO FTRST
1D,2C,3-4,7-10B,11-16		AS104932	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 18"-19" CHAIR WIDTH, 7"-9" SEAT TO FTRST
1D,2D,3-4,7-10B,11-16		AS104933	HIGH MNT ANG ADJ FTRST W/ FRONT ROLLERS 20" CHAIR WIDTH, 7"-9" SEAT TO FTRST



LUGGAGE CARRIER ASSEMBLY


TiLite™ ZR, ZRA, AERO Z, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

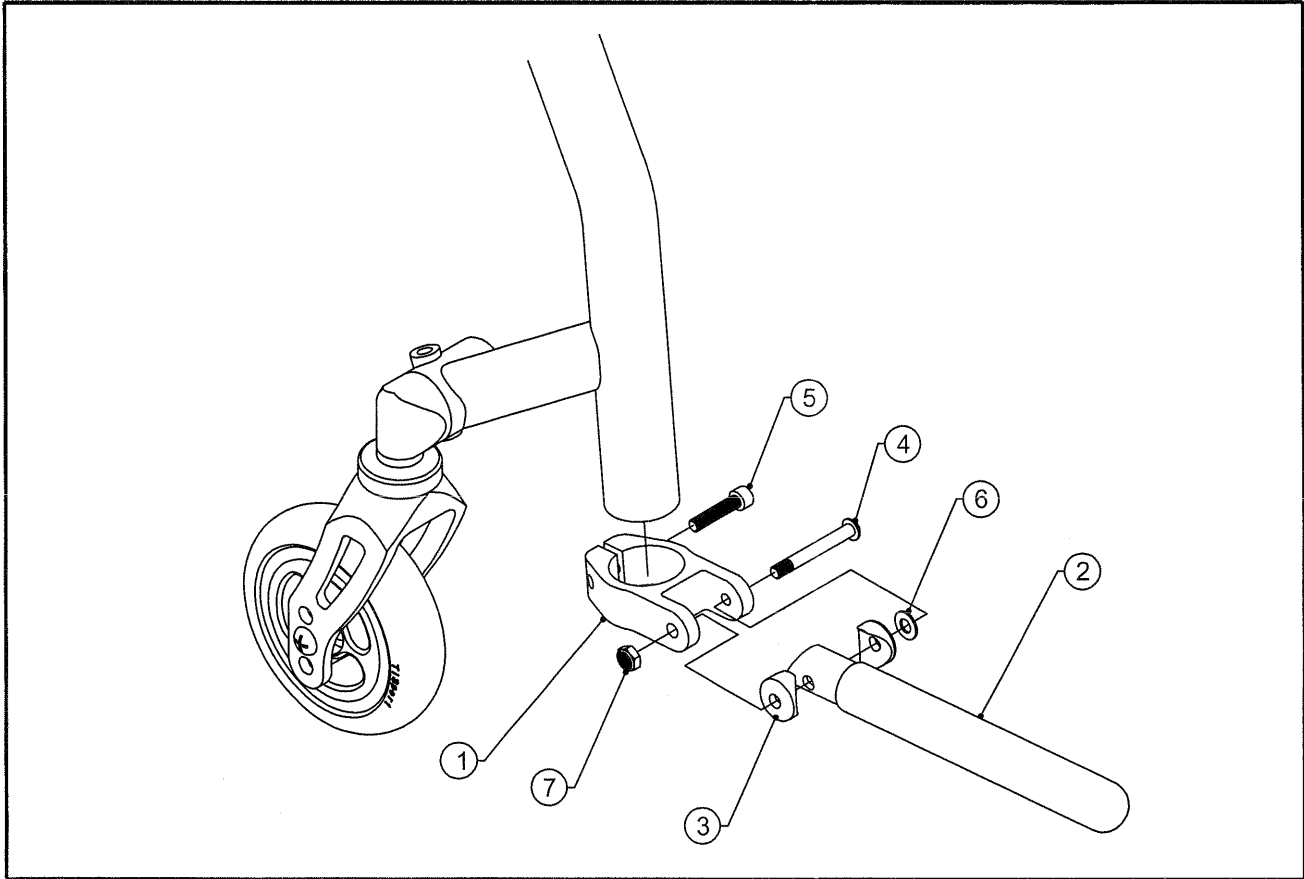

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP102055	LUGGAGE CARRIER CLAMP
2	2	CP101300	LUGGAGE CARRIER TUBE
3	4	HD100119	COVED WASHER 3/4" OD X 1/4" ID X 1/8" THK
4	2	HD100533	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, 1/2" THREAD, BLACK ZINC
5	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
6	2	HD100109	WASHER NYLON .500 OD X .257 ID X .06 THICK
7	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
1-7		AS102355	LUGGAGE CARRIER W/1.25" CLAMP ASSEMBLY, PAIR

LUGGAGE CARRIER ASSEMBLY


TiLite™ TR, YR, TRA, YRA, EVO, BB, X, SX, XC, YG, YGS, Aero X, 2GX, TX, Aero R,
TR (SERIES 2), YR (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2),
Aero X (SERIES 2), 2GX (SERIES 2)

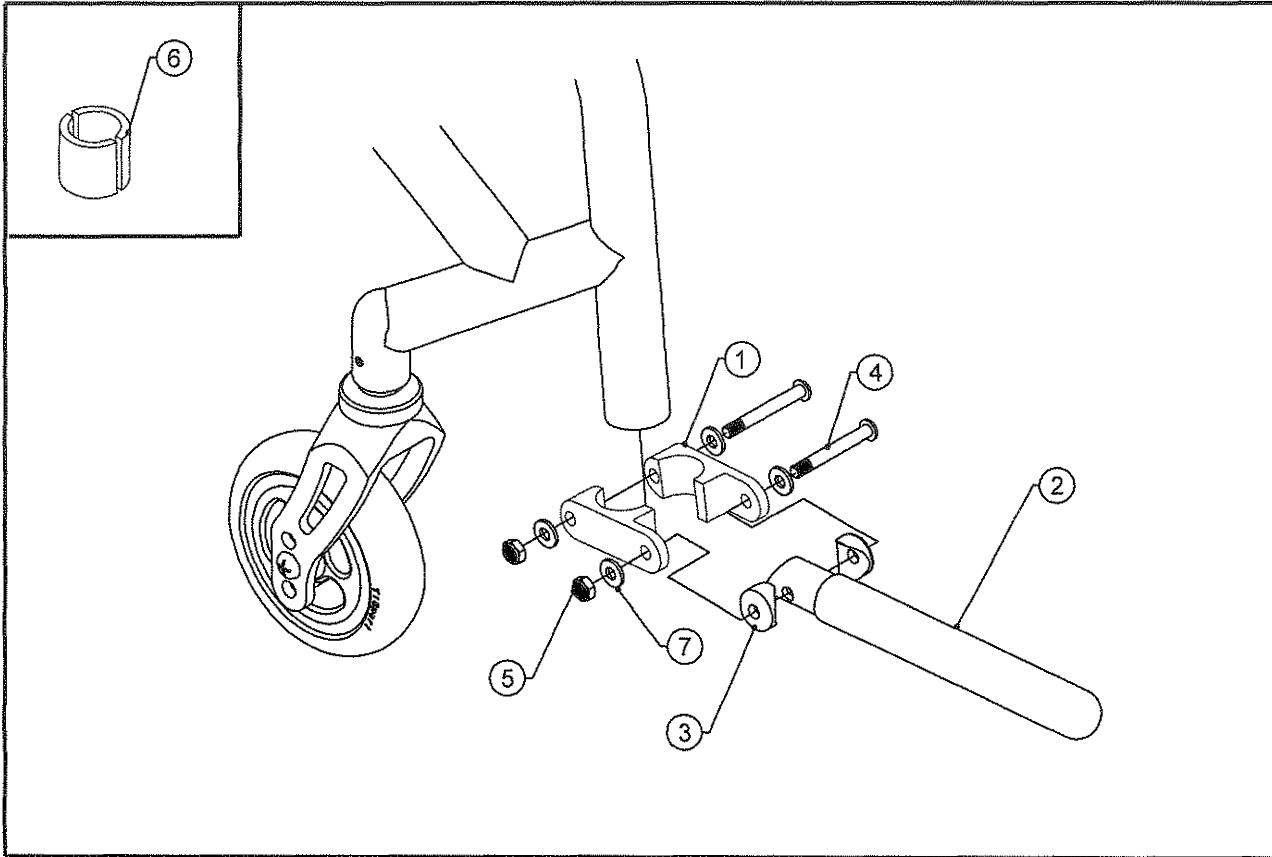

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	4	CP101299	LUGGAGE CARRIER CLAMP
2	2	CP101300	LUGGAGE CARRIER TUBE
3	4	HD100119	COVERED WASHER 3/4" OD X 1/4" ID X 1/8" THK
4	4	HD100533	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, 1/2" THREAD, BLACK ZINC
5	4	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
6	2	CP101304	3/4" TO 1" SPACER LUGGAGE CARRIER (FOR MOUNTING ON 3/4" DIA. TUBE)
7	8	HD100067	NYLON WASHER .500 OD X .265 ID X .031
1-7		AS100973	LUGGAGE CARRIER W/1" CLAMP ASSEMBLY, PAIR

CLAMP-ON PUSH HANDLES

TiLite™ TR, TRA, YR, YRA, ZR, ZRA, EVO, AERO Z, AERO R, TR (SERIES 2), YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2)

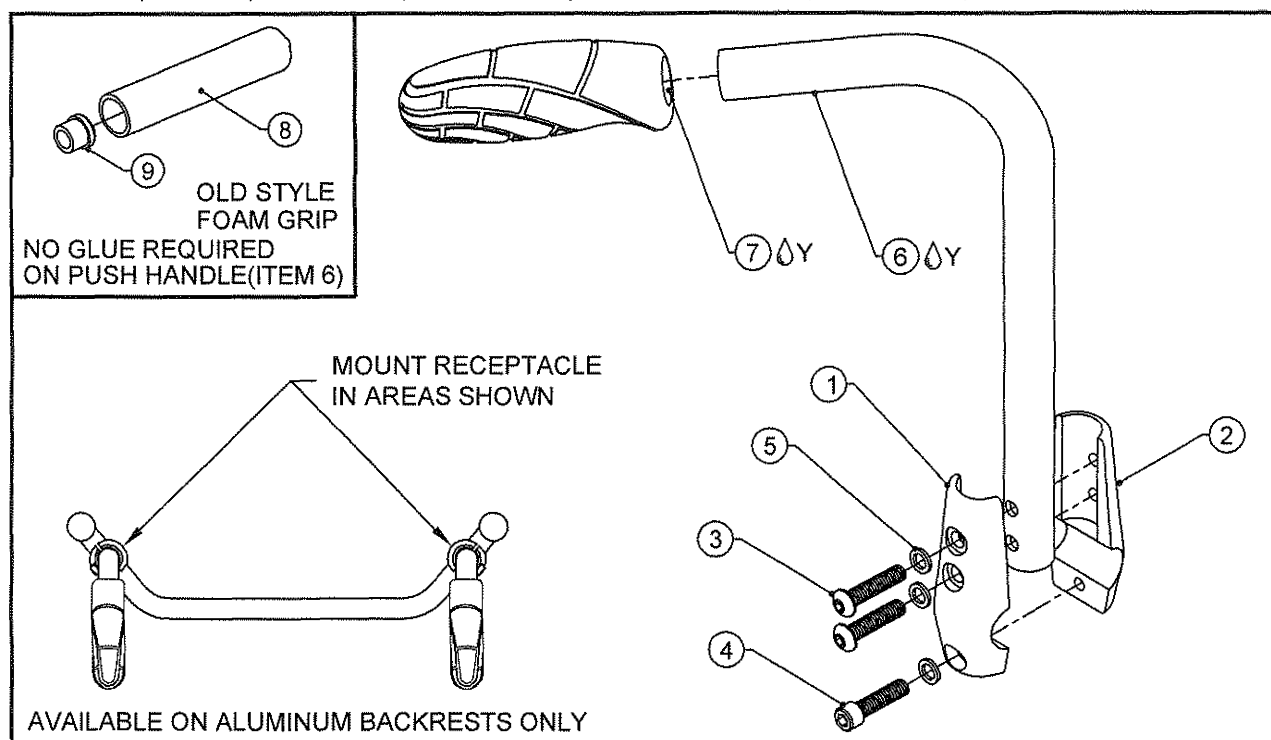

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP102395	PUSH HANDLE CLAMP W/CBORE
2	1	CP102396	PUSH HANDLE CLAMP W/THREADS
3	2	HD100361	1/4-20 X 1-1/8" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
4	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
5	3	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
6A-Y	1	CP103948	CLAMP-ON PUSH HANDLE TALL, RIGHT
6B-Y	1	CP103949	CLAMP-ON PUSH HANDLE TALL, LEFT
6C-Y	1	CP103950	CLAMP-ON PUSH HANDLE MEDIUM, RIGHT
6D-Y	1	CP103951	CLAMP-ON PUSH HANDLE MEDIUM, LEFT
6E-Y	1	CP103952	CLAMP-ON PUSH HANDLE SHORT, RIGHT
6F-Y	1	CP103953	CLAMP-ON PUSH HANDLE SHORT, LEFT
6G-Y	1	CP103954	CLAMP-ON PUSH HANDLE EXTRA SHORT, RIGHT
6H-Y	1	CP103955	CLAMP-ON PUSH HANDLE EXTRA SHORT, LEFT
7-Y	1	AS115918-1	PUSH HANDLE GRIP REPLACEMENT KIT FOR ROUND TUBE, PAIR
8	1	HD100041	FOAM GRIP FOR 3/4" TUBING, .705 I.D. X .187 WALL X 5" LONG
9	1	HD100127	END CAP 3/4" X .083 WALL TUBING
1-5		AS102852	CLAMP-ON PUSH HANDLE CLAMP ASSEMBLY
6A-7		AS104604	CLAMP-ON PUSH HANDLE TALL ASSEMBLY, RIGHT
6B-7		AS104605	CLAMP-ON PUSH HANDLE TALL ASSEMBLY, LEFT
6C-7		AS104606	CLAMP-ON PUSH HANDLE MEDIUM ASSEMBLY, RIGHT
6D-7		AS104607	CLAMP-ON PUSH HANDLE MEDIUM ASSEMBLY, LEFT
6E-7		AS104608	CLAMP-ON PUSH HANDLE SHORT ASSEMBLY, RIGHT
6F-7		AS104609	CLAMP-ON PUSH HANDLE SHORT ASSEMBLY, LEFT
6G-7		AS104610	CLAMP-ON PUSH HANDLE EXTRA SHORT ASSEMBLY, RIGHT
6H-7		AS104611	CLAMP-ON PUSH HANDLE EXTRA SHORT ASSEMBLY, LEFT
1-5,6A-7		AS104612	CLAMP-ON PUSH HANDLE TALL W/CLAMP ASSEMBLY, RIGHT
1-5,6B-7		AS104613	CLAMP-ON PUSH HANDLE TALL W/CLAMP ASSEMBLY, LEFT
1-5,6C-7		AS104614	CLAMP-ON PUSH HANDLE MEDIUM W/CLAMP ASSEMBLY, RIGHT
1-5,6D-7		AS104615	CLAMP-ON PUSH HANDLE MEDIUM W/CLAMP ASSEMBLY, LEFT
1-5,6E-7		AS104616	CLAMP-ON PUSH HANDLE SHORT W/CLAMP ASSEMBLY, RIGHT
1-5,6F-7		AS104617	CLAMP-ON PUSH HANDLE SHORT W/CLAMP ASSEMBLY, LEFT
1-5,6G-7		AS104618	CLAMP-ON PUSH HANDLE EXTRA SHORT W/CLAMP ASSY, RIGHT
1-5,6H-7		AS104619	CLAMP-ON PUSH HANDLE EXTRA SHORT W/CLAMP ASSY, LEFT
1-5,6A,6B-7		AS104620	CLAMP-ON PUSH HANDLE TALL W/CLAMP ASSEMBLY, PAIR
1-5,6C,6D-7		AS104621	CLAMP-ON PUSH HANDLE MEDIUM W/CLAMP ASSEMBLY, PAIR
1-5,6E,6F-7		AS104622	CLAMP-ON PUSH HANDLE SHORT W/CLAMP ASSEMBLY, PAIR
1-5,6G,6H-7		AS104623	CLAMP-ON PUSH HANDLE EXTRA SHORT W/CLAMP ASSY, PAIR

SEE NEXT PAGE FOR ADDITIONAL PUSH HANDLE HEIGHT INFORMATION

CLAMP-ON PUSH HANDLES (CONT.)


TiLite™ TR, TRA, YR, YRA, ZR, ZRA, EVO, AERO Z, AERO R, TR (SERIES 2),
YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2),
TR (SERIES 3), AERO T, TRA (SERIES 2)

PUSH HANDLE AND SEAT BACK STANDARD CONFIGURATIONS

SEAT BACK HEIGHT	8-1/2"	9"	9-1/2"	10"	10-1/2"	11"	11-1/2"	12"	12-1/2"
PUSH HANDLE	EX SHORT	EX SHORT	EX SHORT	EX SHORT	EX SHORT	EX SHORT	SHORT	SHORT	SHORT

SEAT BACK HEIGHT	12-1/2"	13"	13-1/2"	14"	14-1/2"	15"	15-1/2"	16"	16-1/2"
PUSH HANDLE	SHORT	SHORT	SHORT	MEDIUM	MEDIUM	MEDIUM	TALL	TALL	TALL

SEAT BACK HEIGHT	16-1/2"	17"	17-1/2"	18"	18-1/2"	19"	19-1/2"	20"	20-1/2"
PUSH HANDLE	SHORT	SHORT	SHORT	MEDIUM	MEDIUM	MEDIUM	TALL	TALL	TALL

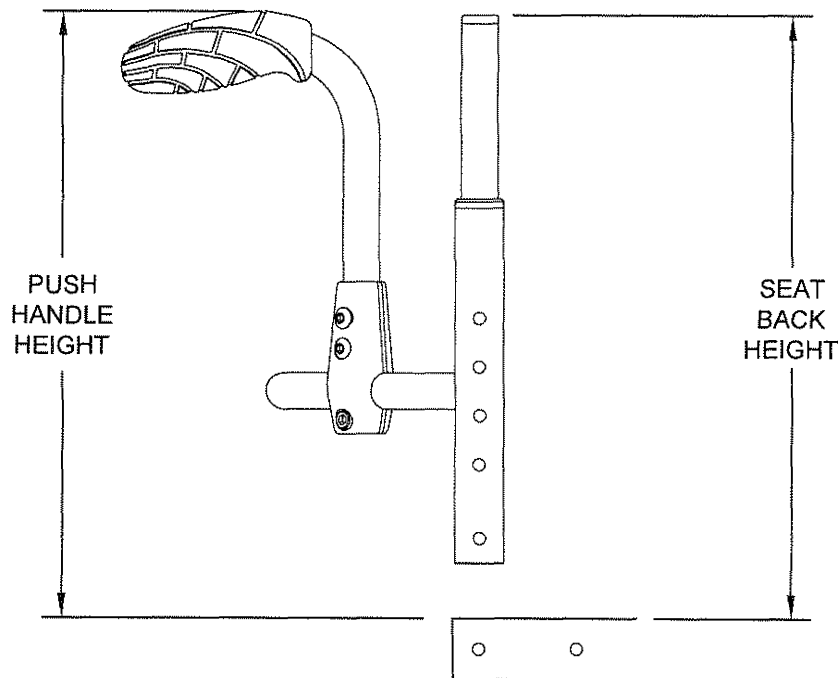


*BACKREST BRACKET REMOVED FOR CLARITY

PUSH HANDLE HEIGHT FROM SEAT

PUSH HANDLE	8-1/2" - 12-1/2" SEAT BACK	12-1/2" - 16-1/2" SEAT BACK	16-1/2" - 20-1/2" SEAT BACK
TALL	15-1/2"	16-1/2"	20-1/2"
MEDIUM	14"	15"	19"
SHORT	12-1/2"	13-1/2"	17-1/2"
EXTRA SHORT	11"	12"	16"

CLAMP-ON PUSH HANDLES

TiLite™ TWIST

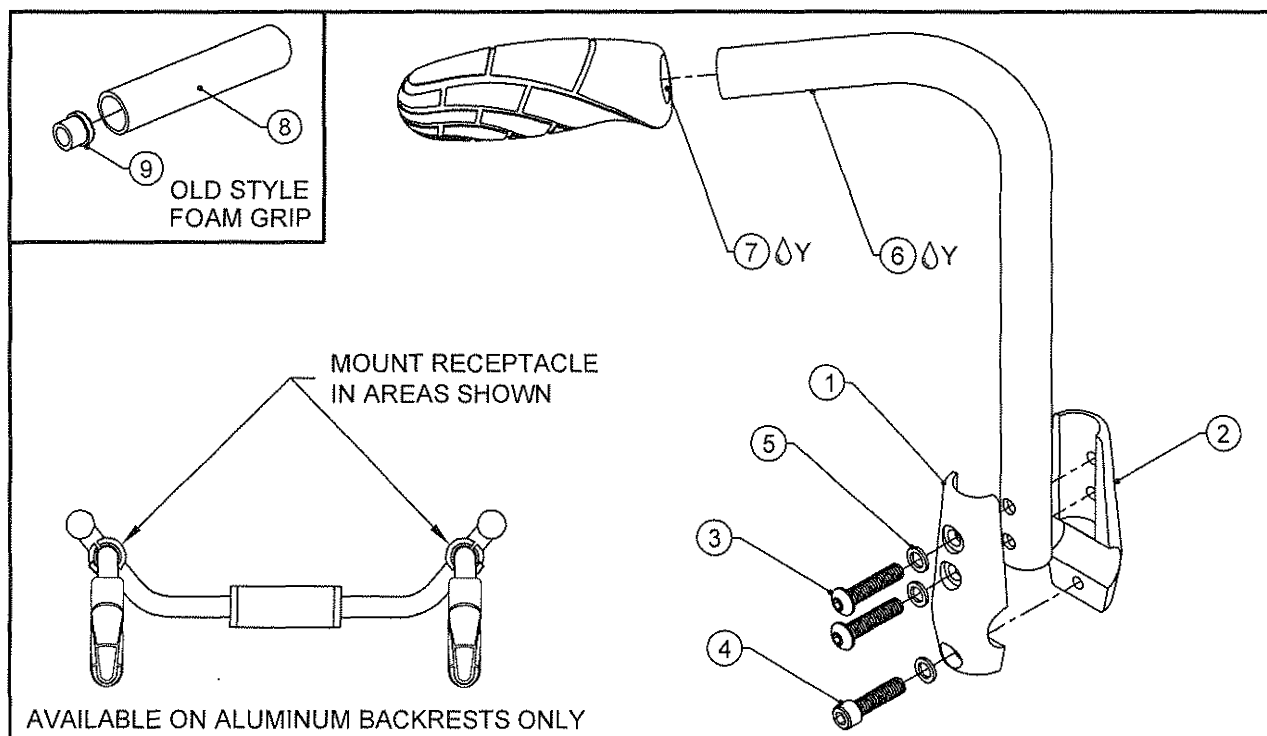

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP102395	PUSH HANDLE CLAMP W/CBORE
2	1	CP102396	PUSH HANDLE CLAMP W/THREADS
3	2	HD100361	1/4-20 X 1-1/8" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
4	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
5	3	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
6A-Y	1	CP103948	CLAMP-ON PUSH HANDLE TALL, RIGHT
6B-Y	1	CP103949	CLAMP-ON PUSH HANDLE TALL, LEFT
6C-Y	1	CP103950	CLAMP-ON PUSH HANDLE MEDIUM, RIGHT
6D-Y	1	CP103951	CLAMP-ON PUSH HANDLE MEDIUM, LEFT
6E-Y	1	CP103952	CLAMP-ON PUSH HANDLE SHORT, RIGHT
6F-Y	1	CP103953	CLAMP-ON PUSH HANDLE SHORT, LEFT
6G-Y	1	CP103954	CLAMP-ON PUSH HANDLE EXTRA SHORT, RIGHT
6H-Y	1	CP103955	CLAMP-ON PUSH HANDLE EXTRA SHORT, LEFT
7-Y	1	AS115918-I	PUSH HANDLE GRIP REPLACEMENT KIT FOR ROUND TUBE, PAIR
8	1	HD100041	FOAM GRIP FOR 3/4" TUBING, .705 I.D. X .187 WALL X 5" LONG
9	1	HD100127	END CAP 3/4" X .083 WALL TUBING
1-5		AS102852	CLAMP-ON PUSH HANDLE CLAMP ASSEMBLY
6A-7		AS104604	CLAMP-ON PUSH HANDLE TALL ASSEMBLY, RIGHT
6B-7		AS104605	CLAMP-ON PUSH HANDLE TALL ASSEMBLY, LEFT
6C-7		AS104606	CLAMP-ON PUSH HANDLE MEDIUM ASSEMBLY, RIGHT
6D-7		AS104607	CLAMP-ON PUSH HANDLE MEDIUM ASSEMBLY, LEFT
6E-7		AS104608	CLAMP-ON PUSH HANDLE SHORT ASSEMBLY, RIGHT
6F-7		AS104609	CLAMP-ON PUSH HANDLE SHORT ASSEMBLY, LEFT
6G-7		AS104610	CLAMP-ON PUSH HANDLE EXTRA SHORT ASSEMBLY, RIGHT
6H-7		AS104611	CLAMP-ON PUSH HANDLE EXTRA SHORT ASSEMBLY, LEFT
1-5,6A-7		AS104612	CLAMP-ON PUSH HANDLE TALL W/CLAMP ASSEMBLY, RIGHT
1-5,6B-7		AS104613	CLAMP-ON PUSH HANDLE TALL W/CLAMP ASSEMBLY, LEFT
1-5,6C-7		AS104614	CLAMP-ON PUSH HANDLE MEDIUM W/CLAMP ASSEMBLY, RIGHT
1-5,6D-7		AS104615	CLAMP-ON PUSH HANDLE MEDIUM W/CLAMP ASSEMBLY, LEFT
1-5,6E-7		AS104616	CLAMP-ON PUSH HANDLE SHORT W/CLAMP ASSEMBLY, RIGHT
1-5,6F-7		AS104617	CLAMP-ON PUSH HANDLE SHORT W/CLAMP ASSEMBLY, LEFT
1-5,6G-7		AS104618	CLAMP-ON PUSH HANDLE EXTRA SHORT W/CLAMP ASSY, RIGHT
1-5,6H-7		AS104619	CLAMP-ON PUSH HANDLE EXTRA SHORT W/CLAMP ASSY, LEFT
1-5,6A,6B-7		AS104620	CLAMP-ON PUSH HANDLE TALL W/CLAMP ASSEMBLY, PAIR
1-5,6C,6D-7		AS104621	CLAMP-ON PUSH HANDLE MEDIUM W/CLAMP ASSEMBLY, PAIR
1-5,6E,6F-7		AS104622	CLAMP-ON PUSH HANDLE SHORT W/CLAMP ASSEMBLY, PAIR
1-5,6G,6H-7		AS104623	CLAMP-ON PUSH HANDLE EXTRA SHORT W/CLAMP ASSY, PAIR

SEE NEXT PAGE FOR ADDITIONAL PUSH HANDLE HEIGHT INFORMATION



CLAMP-ON PUSH HANDLES (CONT.)


TiLite™ TWIST

PUSH HANDLE AND SEAT BACK STANDARD CONFIGURATIONS

8-1/2" - 12-1/2" SEAT BACK HEIGHT	8-1/2"	9"	9-1/2"	10"	10-1/2"	11"	11-1/2"	12"	12-1/2"
PUSH HANDLE	EX SHORT	EX SHORT	SHORT	SHORT	SHORT	MEDIUM	MEDIUM	MEDIUM	TALL

12-1/2" - 16-1/2" SEAT BACK HEIGHT	12-1/2"	13"	13-1/2"
PUSH HANDLE	TALL	TALL	TALL

16-1/2" - 20-1/2" SEAT BACK HEIGHT	16-1/2"	17"	17-1/2"	18"	18-1/2"	19"
PUSH HANDLE	MEDIUM	MEDIUM	MEDIUM	TALL	TALL	TALL

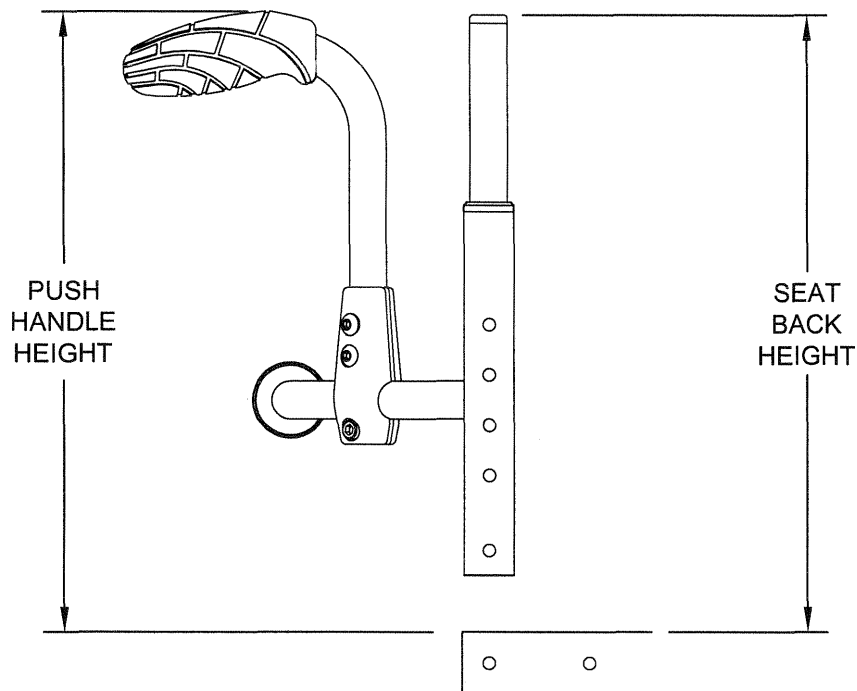


*BACKREST BRACKET REMOVED FOR CLARITY

PUSH HANDLE HEIGHT FROM SEAT TiLite™ TWIST

PUSH HANDLE	8-1/2" - 12-1/2" SEAT BACK	12-1/2" - 16-1/2" SEAT BACK	16-1/2" - 20-1/2" SEAT BACK
TALL	13-1/4"	13-1/4"	19"
MEDIUM	11-3/4"	11-3/4"	17-1/2"
SHORT	10-1/4"	10-1/4"	16"
EXTRA SHORT	8-3/4"	8-3/4"	14-1/2"

BOLT-ON HEIGHT ADJ PUSH HANDLE ASSEMBLY

TiLite™ TR, YR, TRA, YRA, ZR, ZRA, EVO, AERO Z, AERO R, TR (SERIES 2), YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TWIST, TR (SERIES 3), AERO T, TRA (SERIES 2)

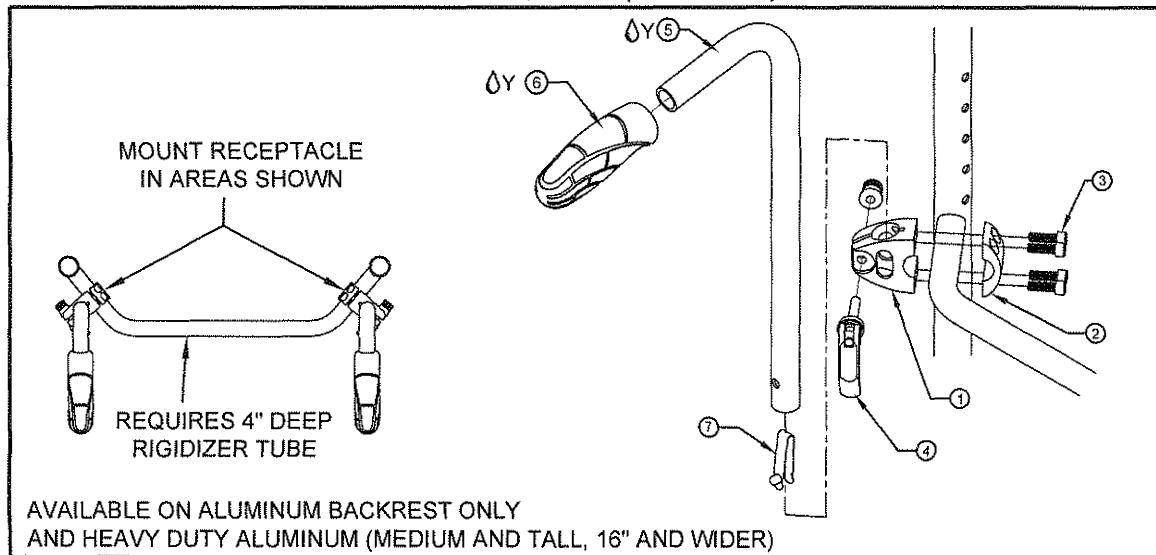

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP106975	TELESCOPING PUSH HANDLE CLAMP REAR
2	1	CP106974	TELESCOPING PUSH HANDLE CLAMP FRONT
3	4	HD100008	1/4-20 X 5/8 LONG SOCKET HEAD CAP SCREW
4	1	HD100702	QUICK RELEASE CAM LOCK
5A-Y	1	CP102380	ADJUSTABLE HEIGHT PUSH HANDLE
5B-Y	1	CP103279	SHORT ADJUSTABLE HEIGHT PUSH HANDLE
6-Y	1	CP102565	TiLite™ PUSH HANDLE GRIP
7	1	HD100155	SINGLE END SNAP BUTTON
1-4		AS101336	TELESCOPING PUSH HANDLE CLAMP ASSEMBLY
5A-7		AS102826	ADJUSTABLE HEIGHT PUSH HANDLE ASSEMBLY
5B-7		AS111098	SHORT ADJUSTABLE HEIGHT PUSH HANDLE ASSEMBLY(FOR TALL BACKREST)
1-4, 5A-7		AS101337	TELESCOPING PUSH HANDLE ASSEMBLY, PAIR
1-4, 5B-7		AS111099	SHORT TELESCOPING PUSH HANDLE ASSEMBLY, PAIR(FOR TALL BACKREST)

* AS115918-I PUSH HANDLE GRIP REPLACEMENT KIT FOR ROUND TUBE, PAIR

TR/TRC/YR/TRA/EVO HEIGHT RANGE (TOP OF SEAT TO TOP OF HANDLE)

	10"-15" WIDTH	16"-20" WIDTH
8" - 12" STANDARD BACKREST	7.250" - 23.375"	7.250" - 23.375"
12" - 16" STANDARD BACKREST	8.375" - 24.500"	7.375" - 23.500"
16" - 20" STANDARD BACKREST	12.375" - 24.500"	11.375" - 23.500"

ZR/ZRA/AERO Z HEIGHT RANGE (TOP OF SEAT TO TOP OF HANDLE)

	10"-15" WIDTH	16"-20" WIDTH
8.5" - 12.5" STANDARD BACKREST	7.625" - 23.750"	7.625" - 23.750"
12.5" - 16.5" STANDARD BACKREST	8.750" - 24.875"	7.750" - 23.875"
16.5" - 20.5" STANDARD BACKREST	12.750" - 24.875"	11.750" - 23.875"


TWIST HEIGHT RANGE (TOP OF SEAT TO TOP OF HANDLE)

8.5" - 12.5" STANDARD BACKREST	5.375" - 21.500"
12.5" - 16.5" STANDARD BACKREST	5.375" - 21.500"
16.5" - 20.5" STANDARD BACKREST	11.250" - 23.375"

MAXIMUM ADJUSTABILITY OF TELESCOPING PUSH HANDLE IS 16.125"
TELESCOPING PUSH HANDLES MAY ONLY BE ATTACHED TO AN
ALUMINUM BACKREST WITH A 4" DEEP RIGIDIZER BAR

ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR

TiLite™ TX, X, SX, Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

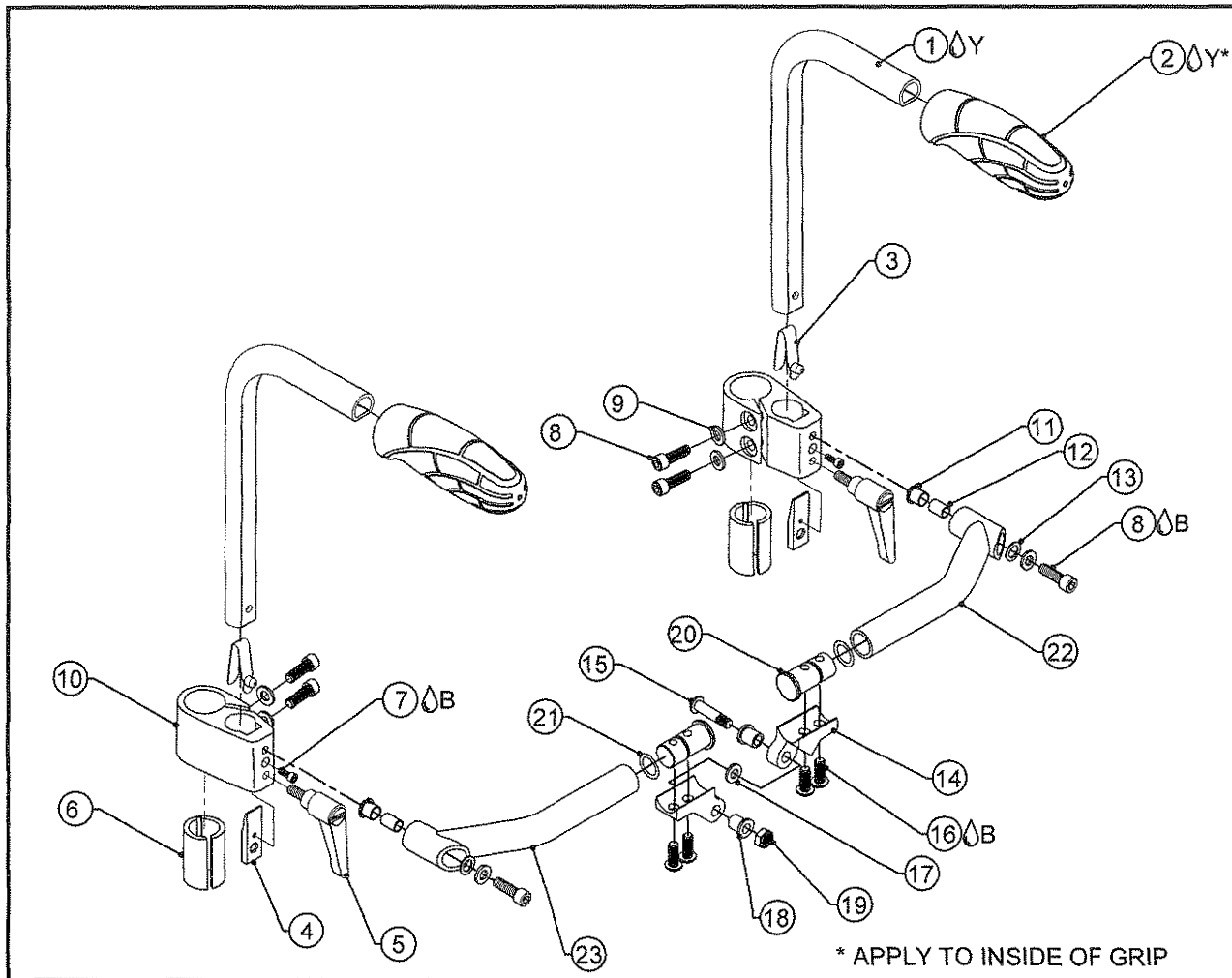

ITEM	QUANTITY	PART NO.	DESCRIPTION
10Y	1	CP102912	ADJUSTABLE HEIGHT PUSH HANDLE - D SHAPE
20Y	1	AS115919-I	PUSH HANDLE GRIP REPLACEMENT KIT FOR D-SHAPED TUBE, PAIR
3	1	HD100155	SINGLE END SNAP BUTTON
4	2	CP103197	SERRATED ADJ HT PUSH HANDLE PLATE
5	2	HD100428	1/4-20 X .78" STUD ADJUSTABLE HANDLE, BLACK
6	2	CP102593	Ø1.00 TO Ø.75 SPLIT SLEEVE
7B	2	HD100419	6-32 X 3/8" SHCS, BLACK ZINC
8B	6	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
9	6	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THK
10	2	CP102543	ADJ HT PUSH HANDLE W/ FOLDING BAR RECEPTAL
11	2	HD100475	FLANGED NYLINER BEARING 5/16" ID, 5/16" LG
12	2	CP100243	BACKREST LATCH SPACER
13	2	HD100477	WASHER NYLON .316 ID X .500 OD X .031 THK, BLACK
14	2	CP103230	BOLT ON PIVOT FOR FOLDING BACKREST BAR
15	1	HD100484	BUTTON HEAD CAP SCREW 1/4-28 X 1 1/4 BLACK ZINC
16B	4	HD100319	1/4-20 X 5/8" BUTTON HEAD CAP SCREW, ALLOY STEEL, CLEAR ZINC
17	1	HD100483	WASHER NYLON .315 ID X .507 OD X .062 THK, BLACK
18	2	HD100474	FLANGED SCREW ISULATOR .260 ID X .375 LG
19	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
20	2	CP100108	BACKREST TUBE EXTENSION PLUG
21	2	HD100043	O-RING 1/2 I.D. X 5/8 O.D. BUNA-N TRADE SIZE: #014
22	2	TABLE ONE	CENTER FOLDING BACKREST HALF, RIGHT (SEE NEXT PAGE)
23	1	TABLE TWO	CENTER FOLDING BACKREST HALF, LEFT (SEE NEXT PAGE)
1-3		AS103169	PUSH HANDLE, D-SHAPE
1-23		TABLE THREE	ADJUSTABLE HEIGHT CLAMP-ON PUSH HANDLE (SEE NEXT PAGE) W/FOLDING BACKREST BAR ASSY

ASSEMBLIES AND TABLES SHOWN ON NEXT PAGE

ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR (CONT.)

TiLite™ TX, X, SX, Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE ONE - CENTER FOLDING BACKREST HALF, RIGHT

SEAT WIDTH	12"	13"	14"	15"	16"	17"	18"	19"	20"
PART NUMBER	CP103247	CP103248	CP103249	CP103250	CP103251	CP103252	CP103253	CP103254	CP103255

TABLE TWO - CENTER FOLDING BACKREST HALF, LEFT

SEAT WIDTH	12"	13"	14"	15"	16"	17"	18"	19"	20"
PART NUMBER	CP103256	CP103257	CP103258	CP103259	CP103260	CP103261	CP103262	CP103263	CP103264


TABLE THREE - ADJUSTABLE HEIGHT CLAMP-ON PUSH HANDLE W/FOLDING BACKREST BAR ASSY

SEAT WIDTH	12"	13"	14"	15"	16"	17"	18"	19"	20"
PART NUMBER	AS102830	AS102831	AS102832	AS102833	AS102834	AS102835	AS102836	AS102837	AS102838



FOLDING BACKREST BAR ASSEMBLY

TiLite™ TX, X, SX, Aero X, 2GX, , Aero X (SERIES 2), 2GX (SERIES 2)

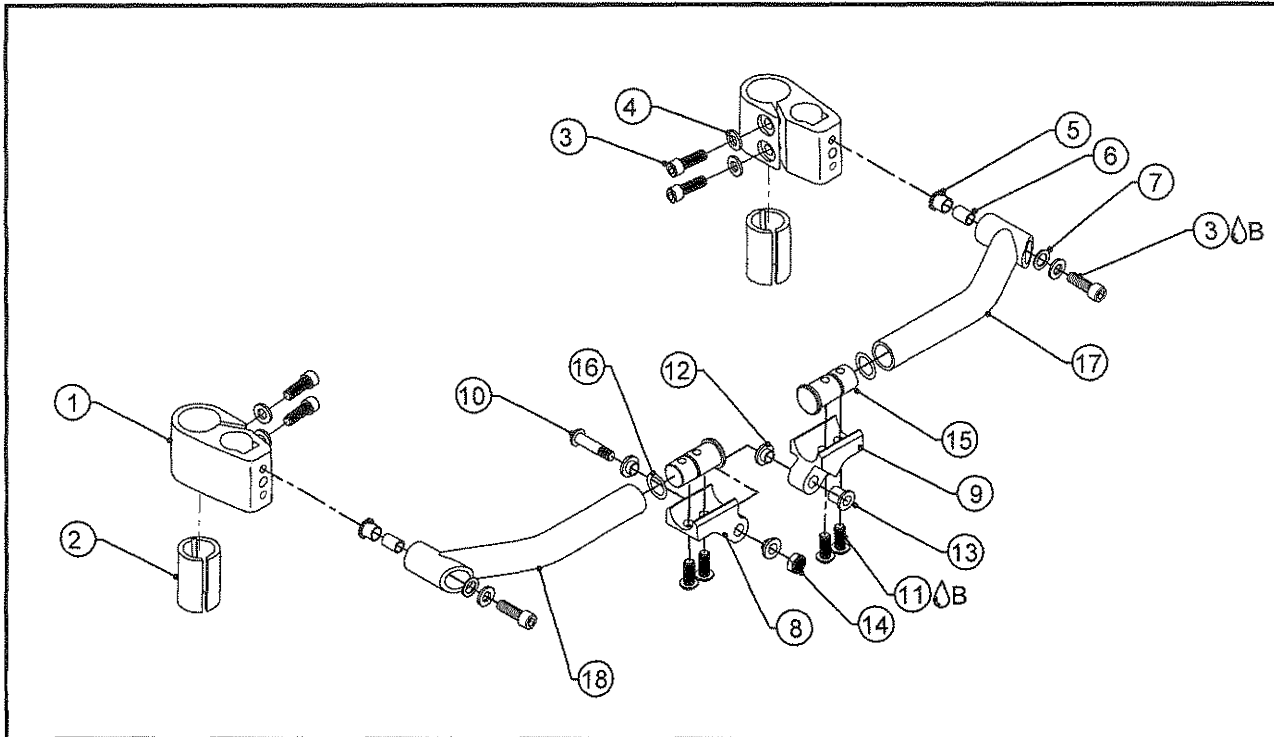

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP102543	ADJ HT PUSH HANDLE W/ FOLDING BAR RECEPTICAL
2	2	CP102593	Ø1.00 TO Ø.75 SPLIT SLEEVE
3 ^{AB}	6	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
4	6	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
5	2	HD100475	FLANGED NYLINER BEARING 5/16" ID, 5/16" LG
6	2	CP100243	BACKREST LATCH SPACER
7	2	HD100477	WASHER NYLON .316 ID X .500 OD X .031 THK, BLACK
8	1	CP104126	HD FOLDING BACKREST BAR PIVOT A
9	1	CP104127	HD FOLDING BACKREST BAR PIVOT B
10	1	HD100564	1/4-28 X 1-3/8" W/ 3/8" THREAD BUTTON HEAD CAP SCREW, BLACK
11 ^{AB}	4	HD100319	1/4-20 X 5/8" BUTTON HEAD CAP SCREW, ALLOY STEEL, CLEAR ZINC
12	3	HD100482	FLANGED SCREW ISULATOR .260 ID X .125 LG
13	1	HD100474	FLANGED SCREW ISULATOR .260 ID X .375 LG
14	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
15	2	CP100108	BACKREST TUBE EXTENSION PLUG
16	2	HD100043	O-RING 1/2 I.D. X 5/8 O.D. BUNA-N TRADE SIZE: #014
17	1	TABLE ONE	CENTER FOLDING BACKREST HALF, RIGHT
18	1	TABLE TWO	CENTER FOLDING BACKREST HALF, LEFT
1-18		TABLE THREE	FOLDING BACKREST BAR ASSEMBLY

TABLE ONE - CENTER FOLDING BACKREST HALF, RIGHT

SEAT WIDTH	12"	13"	14"	15"	16"	17"	18"	19"	20"	21"	22"
PART NUMBER	CP103247	CP103248	CP103249	CP103250	CP103251	CP103252	CP103253	CP103254	CP103255	CP103927	CP103928

TABLE TWO - CENTER FOLDING BACKREST HALF, LEFT

SEAT WIDTH	12"	13"	14"	15"	16"	17"	18"	19"	20"	21"	22"
PART NUMBER	CP103256	CP103257	CP103258	CP103259	CP103260	CP103261	CP103262	CP103263	CP103264	CP103925	CP103926


TABLE THREE - FOLDING BACKREST BAR ASSEMBLY

SEAT WIDTH	12"	13"	14"	15"	16"	17"	18"	19"	20"	21"	22"
PART NUMBER	AS103170	AS103171	AS103172	AS103173	AS103174	AS103175	AS103176	AS103177	AS103178	AS110473	AS110474



ADJUSTABLE HEIGHT PUSH HANDLES


TiLite™ TX, X, SX, Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

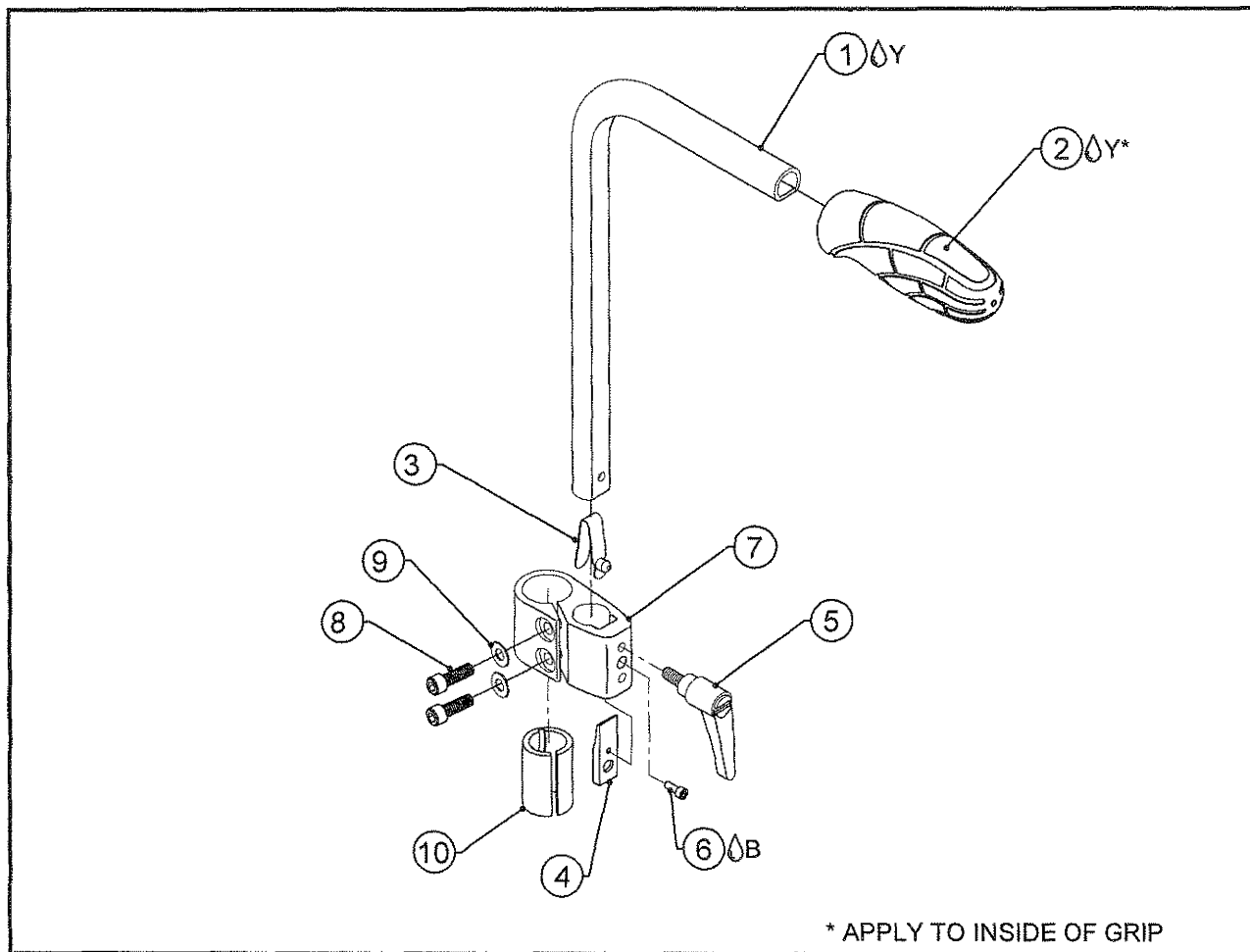

ITEM	QUANTITY	PART NO.	DESCRIPTION
1 Δ Y	1	CP102912	ADJUSTABLE HEIGHT PUSH HANDLE - D SHAPE
2 Δ Y	1	AS115919-I	PUSH HANDLE GRIP REPLACEMENT KIT FOR D-SHAPED TUBE, PAIR
3	1	HD100155	SINGLE END SNAP BUTTON
4	2	CP103197	SERRATED ADJ HT PUSH HANDLE PLATE
5	2	HD100428	1/4-20 X .78" STUD ADJUSTABLE HANDLE, BLACK
6 Δ B	2	HD100419	6-32 X 3/8" SHCS, BLACK ZINC
7	2	CP102543	ADJ HT PUSH HANDLE W/ FOLDING BAR RECEPTICAL
8	4	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
9	4	HD100056	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK
10	2	CP102593	Ø1.00 TO Ø.75 SPLIT SLEEVE
1-3		AS103169	PUSH HANDLE, D-SHAPE
1-10		AS102839	ADJUSTABLE HEIGHT CLAMP-ON PUSH HANDLES
1-9		AS104627	ADJUSTABLE HEIGHT CLAMP-ON PUSH HANDLES FOR TX

CENTER MOUNT PUSH HANDLE

TiLite™ TWIST

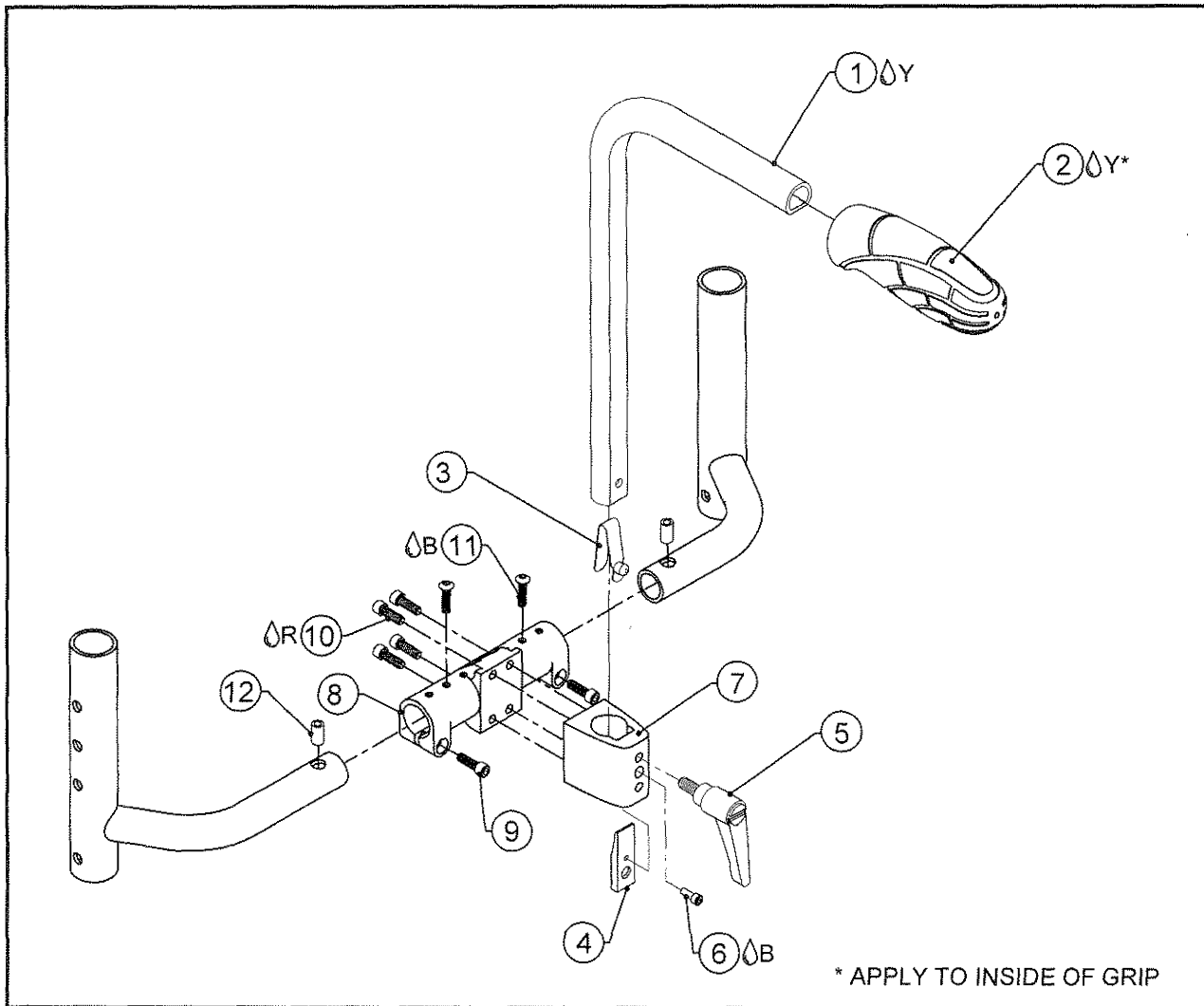


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A Δ Y	1	CP102912	ADJUSTABLE HEIGHT PUSH HANDLE - D SHAPE 9.25"
1B Δ Y	1	CP105121	ADJUSTABLE HEIGHT PUSH HANDLE - D SHAPE 14.25"
1C Δ Y	1	CP105915	ADJUSTABLE HEIGHT PUSH HANDLE - D SHAPE 19.25"
2 Δ Y	1	AS115919-I	PUSH HANDLE GRIP REPLACEMENT KIT FOR D-SHAPED TUBE, PAIR
3	1	HD100155	SINGLE END SNAP BUTTON
4	1	CP103197	SERRATED ADJ HT PUSH HANDLE PLATE
5	1	HD100428	1/4-20 X .78" STUD ADJUSTABLE HANDLE, BLACK
6 Δ B	1	HD100419	6-32 X 3/8" SHCS, BLACK ZINC
7	1	CP105116	PUSH MOUNT
8	1	CP105115	MOUNT CLAMP
9	2	HD100427	10-32 X 3/4" SHCS, BLACK ZINC
10 Δ R	4	HD100679	10-32 X 1/2 SOCKET HEAD CAP SCREW, BLACK ZINC
11 Δ B	2	HD100680	10-32 X 7/8 BUTTON HEAD CAP SCREW, BLACK ZINC
12	2	CP107122	STROLLER HANDLE SPACER
1A-3		AS103169	PUSH HANDLE, D-SHAPE 9.25"
1B-3		AS111884	PUSH HANDLE, D-SHAPE 14.25"
1C-3		AS111992	PUSH HANDLE, D-SHAPE 19.25"
1A-12		AS111882	CENTER MOUNT PUSH HANDLE ASSY 9.25"
1B-12		AS111883	CENTER MOUNT PUSH HANDLE ASSY 14.25"
1C-12		AS111993	CENTER MOUNT PUSH HANDLE ASSY 19.25"

TWIST HEIGHT RANGE: TOP OF SEAT TO TOP OF HANDLE

8.5" - 12.5" STANDARD BACKREST	5.375" - 23.500"
12.5" - 16.5" STANDARD BACKREST	5.375" - 23.500"
16.5" - 20.5" STANDARD BACKREST	11.250" - 25.375"

CENTER MOUNT STROLLER HANDLE


TiLite™ TWIST

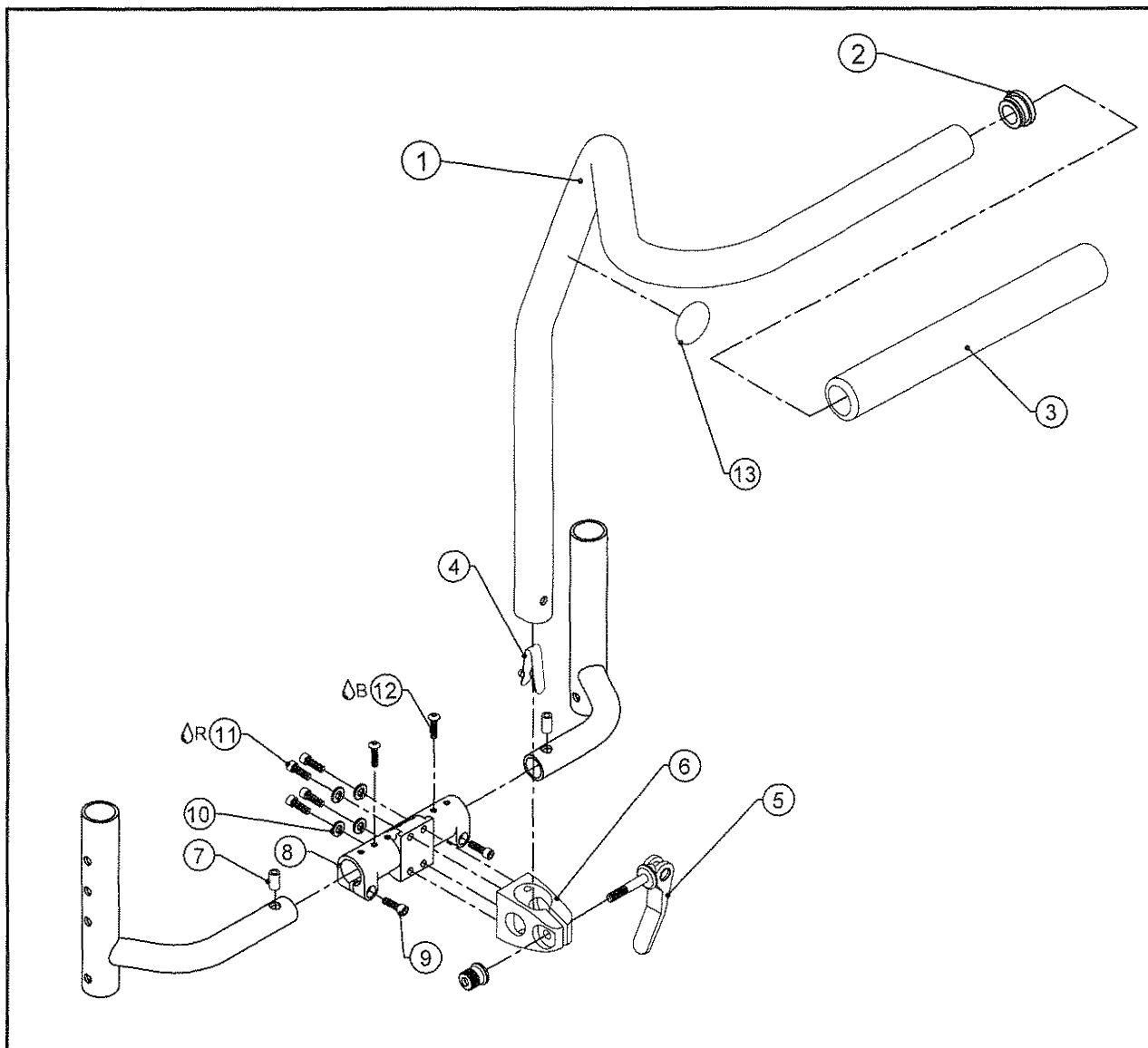

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP107124	STROLLER HANDLE TUBE
2	1	HD100669	TWIST SIDEFRAME END CAP
3	1	HD100112	FOAM GRIP FOR 1" TUBING, .940 I.D. X .125 WALL X 12" LONG
4	1	HD100155	SINGLE END SNAP BUTTON
5	1	HD100702	QUICK RELEASE CAM LOCK
6	1	CP107123	STROLLER HANDLE CLAMP
7	1	CP107122	STROLLER HANDLE SPACER
8	1	CP105115	MOUNT CLAMP
9	2	HD100427	10-32 X 3/4" SHCS, BLACK ZINC
10	4	HD100269	10-32 SPLIT LOCK WASHER
11 R	4	HD100679	10-32 X 1/2" SHCS, BLACK ZINC
12 B	2	HD100680	10-32 X 7/8" BHCS, BLACK ZINC
13	1	CP107227	DECAL - STROLLER HANDLE WARNING GRAPHIC
1-13		AS115910-1	STROLLER HANDLE ASSEMBLY

YG/YGS TIE DOWN PACKAGE


TiLite™ YG, YGS

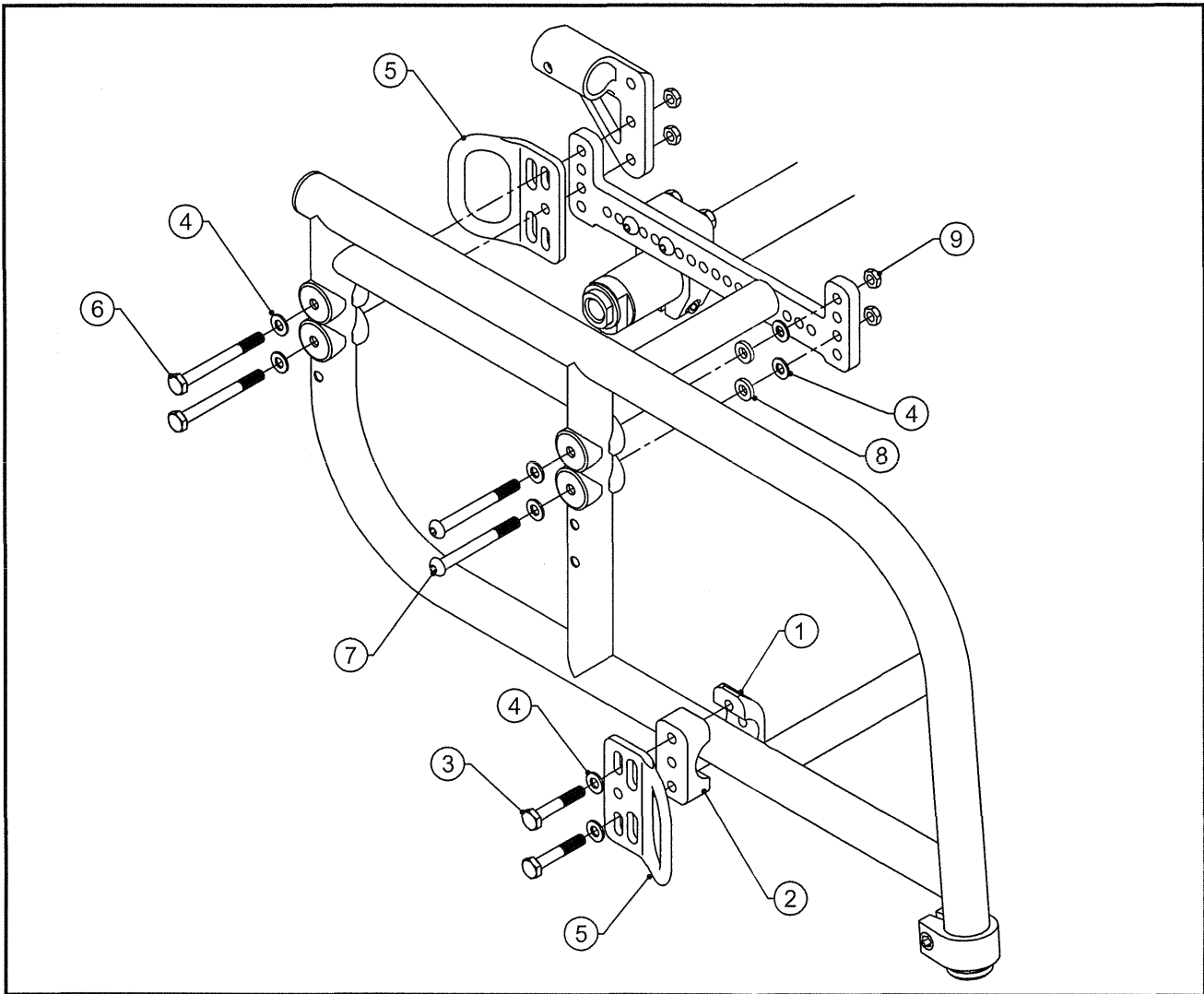

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP100519	C CLAMP CASTER
2	2	CP100520	U CLAMP CASTER
3	4	HD100380	1/4-20 X 1-1/2" HEX CAP SCREW, GRADE 5, ZINC PLATE
4	16	HD100022	1/4 SAE FLAT WASHER, CLEAR ZINC PLATED
5	4	CP102513	TIE DOWN ANCHOR MOUNT
6A	4	HD100381	1/4-28 X 2-1/2" HEX CAP SCREW, GRADE 5, ZINC PLATE
6B	4	HD100457	1/4-28 X 2-3/4" HEX CAP SCREW, GRADE 8, ZINC PLATE
7	4	HD100105	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BUTTON HEAD CAP SCREW
8	4	HD100231	NYLON WASHER, .260 ID X .750 OD X .125 THICK BLACK
9	8	HD100021	1/4-28 X .218 TALL NYLOCK LOCKNUT, CLEAR ZINC PLATED
1-6A,7-9		AS102925	YG/YGS TIE DOWN PACKAGE
1-6B,7-9		AS103394	YG/YGS TIE DOWN PACKAGE FOR USE WITH STROLLER HANDLES AND ANTI-TIPS

TRANSPORT BRACKET OPTION

TiLite™ ZR (SERIES 2)

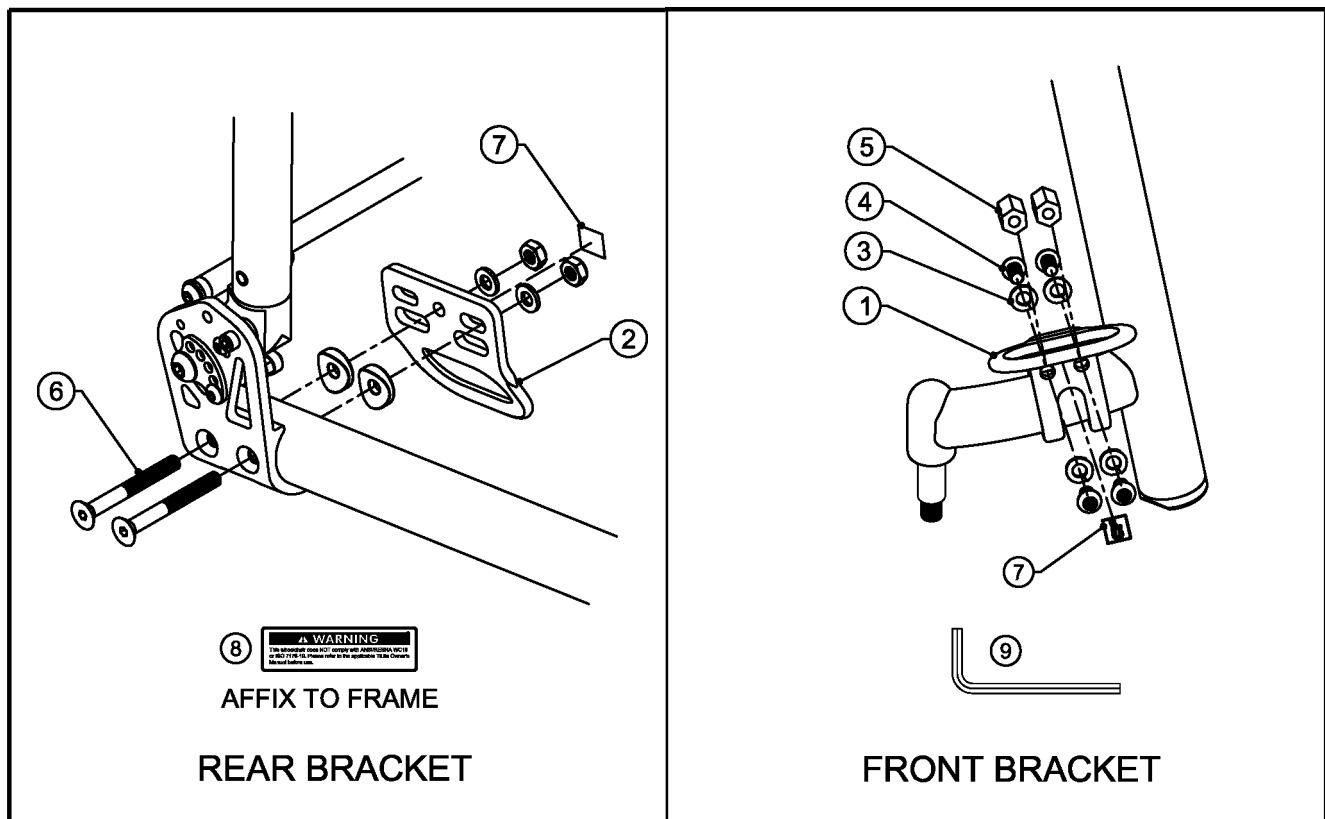


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	CP108286	COUPLING NUT 1/4-20 X .690"
6	4	HD100606	1/4-28 X 2-1/4" FLAT HEAD CAP SCREW BLACK ZINC
7	4	CP103109	DECAL - TRANSPORT HOOK
8	1	CP105025	DECAL - TRANSPORT BRACKET
9	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*10	1	DC100889	TRANSPORT BRACKET FIELD ASSEMBLY GUIDE
*11	1	DC100896	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*12	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-8		AS115935	TRANSPORT BRACKET PACKAGE
1-12		AS115939-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT

* NOT SHOWN

TRANSPORT BRACKET OPTION FOR USE W/ ARMRESTS

TiLite™ ZR (SERIES 2)

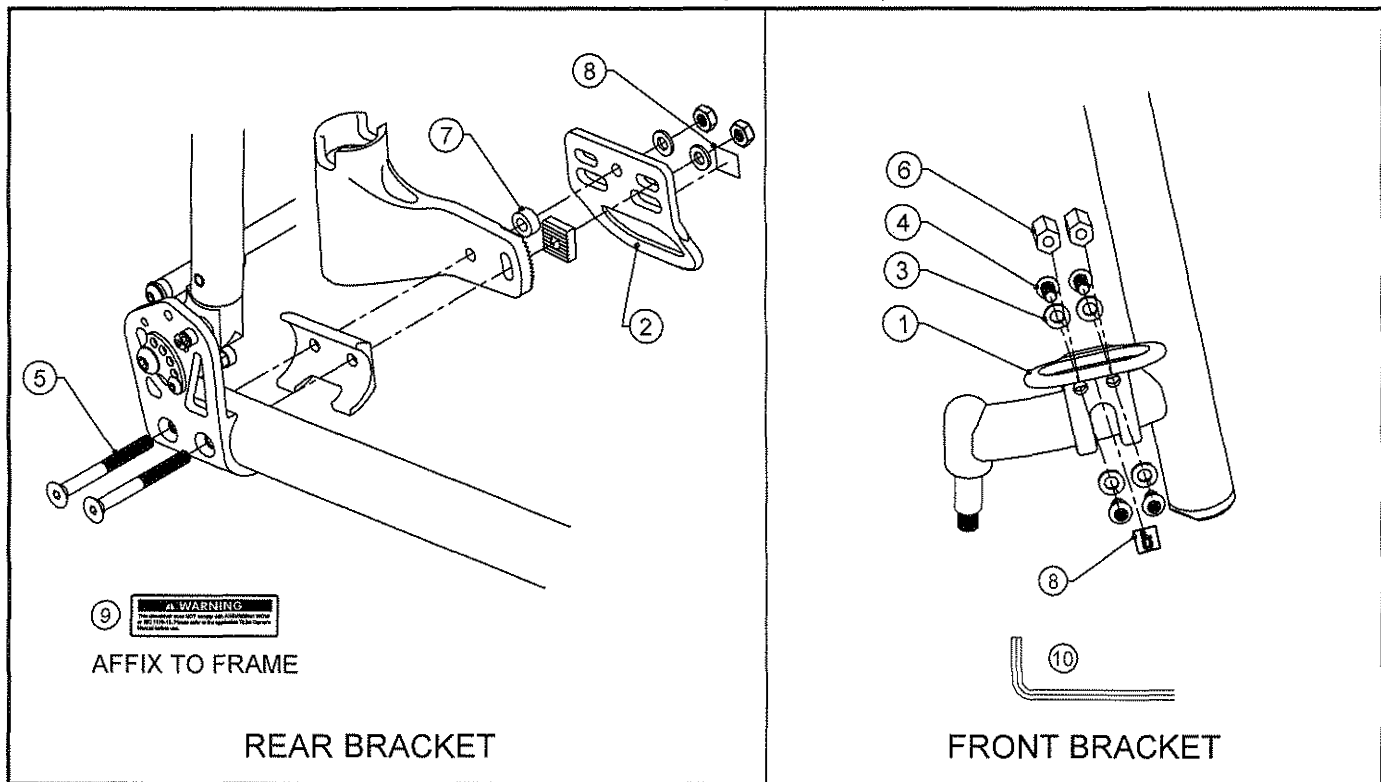


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100699	1/4-20 X 5/8 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	HD100647	1/4-28 X 2-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	4	CP104972	COUPLING NUT 1/4-20 X 1-1/16"
7	2	HD100394	SPACER
8	4	CP103109	DECAL - TRANSPORT HOOK
9	1	CP105025	DECAL - TRANSPORT BRACKET
10	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*11	1	DC100889	TRANSPORT BRACKET FIELD ASSEMBLY GUIDE
*12	1	DC100896	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*13	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-9		AS115908	TRANSPORT BRACKET PACKAGE FOR USE W/ ARMRESTS
1-13		AS115939-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT

*NOT SHOWN

TRANSIT TIE-DOWN OPTION


TiLite™ ZRA (SERIES 2), DEPTH ADJUSTABLE ZRA (SERIES 2),
AERO Z (TITANIUM FRAME), DEPTH ADJUSTABLE AERO Z (TITANIUM FRAME)

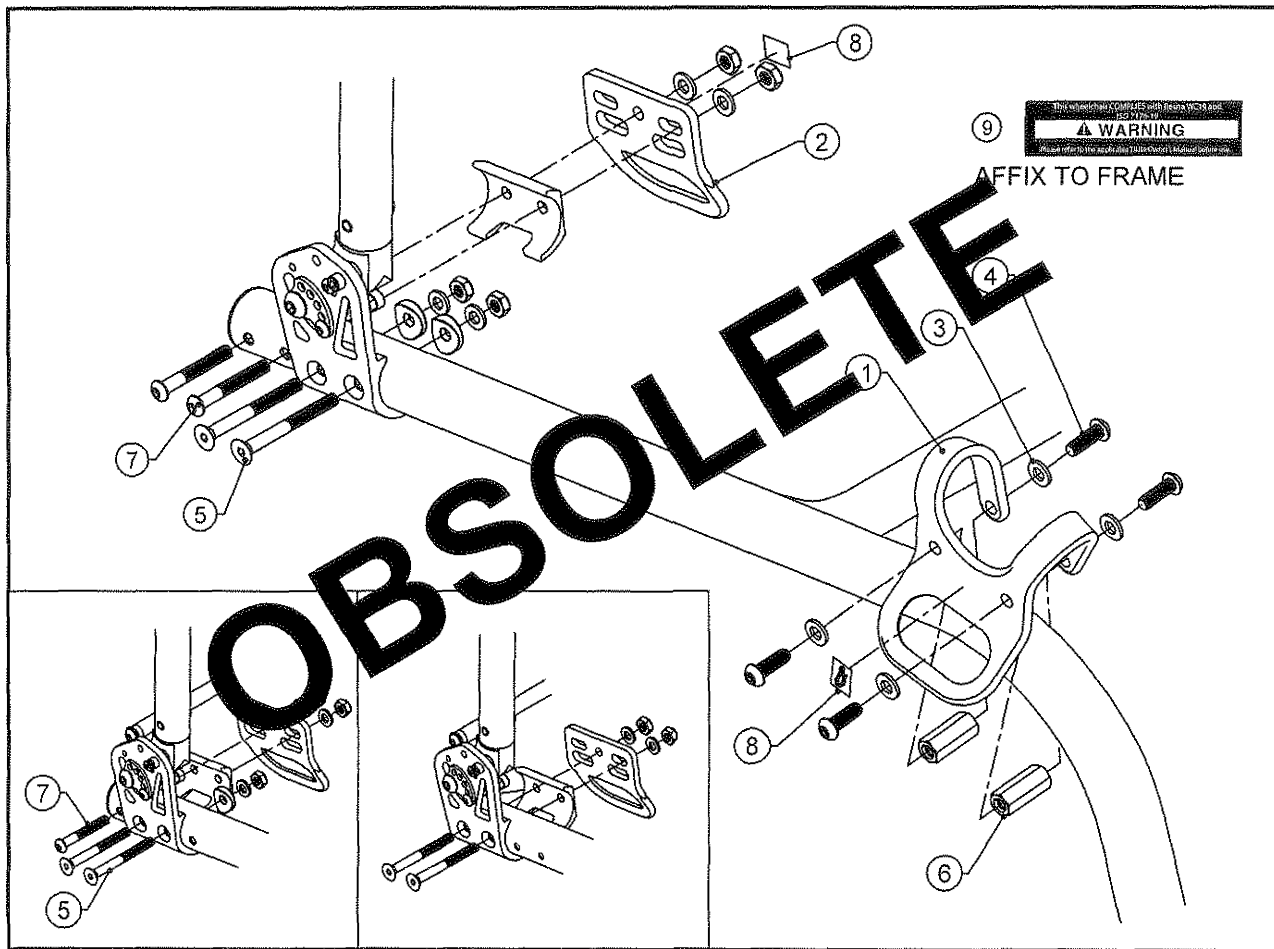

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP104971	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100646	1/4-20 X 3/4" BUTTON HEAD CAP SCREW, ALLOY STEEL, BLACK ZINC
5	4	HD100606	1/4-28 X 2-1/4" FLAT HEAD CAP SCREW BLACK ZINC
6	4	CP104972	COUPLING NUT 1/4-20 X 1-1/16"
7	4	HD100655	1/4-28 X 2-1/2" BUTTON HEAD CAP SCREW, BLACK ZINC
8	4	CP103109	DECAL - TRANSPORT HOOK
9	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
1-9		AS111232	TRANSIT TIE-DOWN PACKAGE

TRANSIT TIE-DOWN OPTION FOR USE W/ ARMRESTS


TiLite™ ZRA (SERIES 2), AERO Z (TITANIUM FRAME)

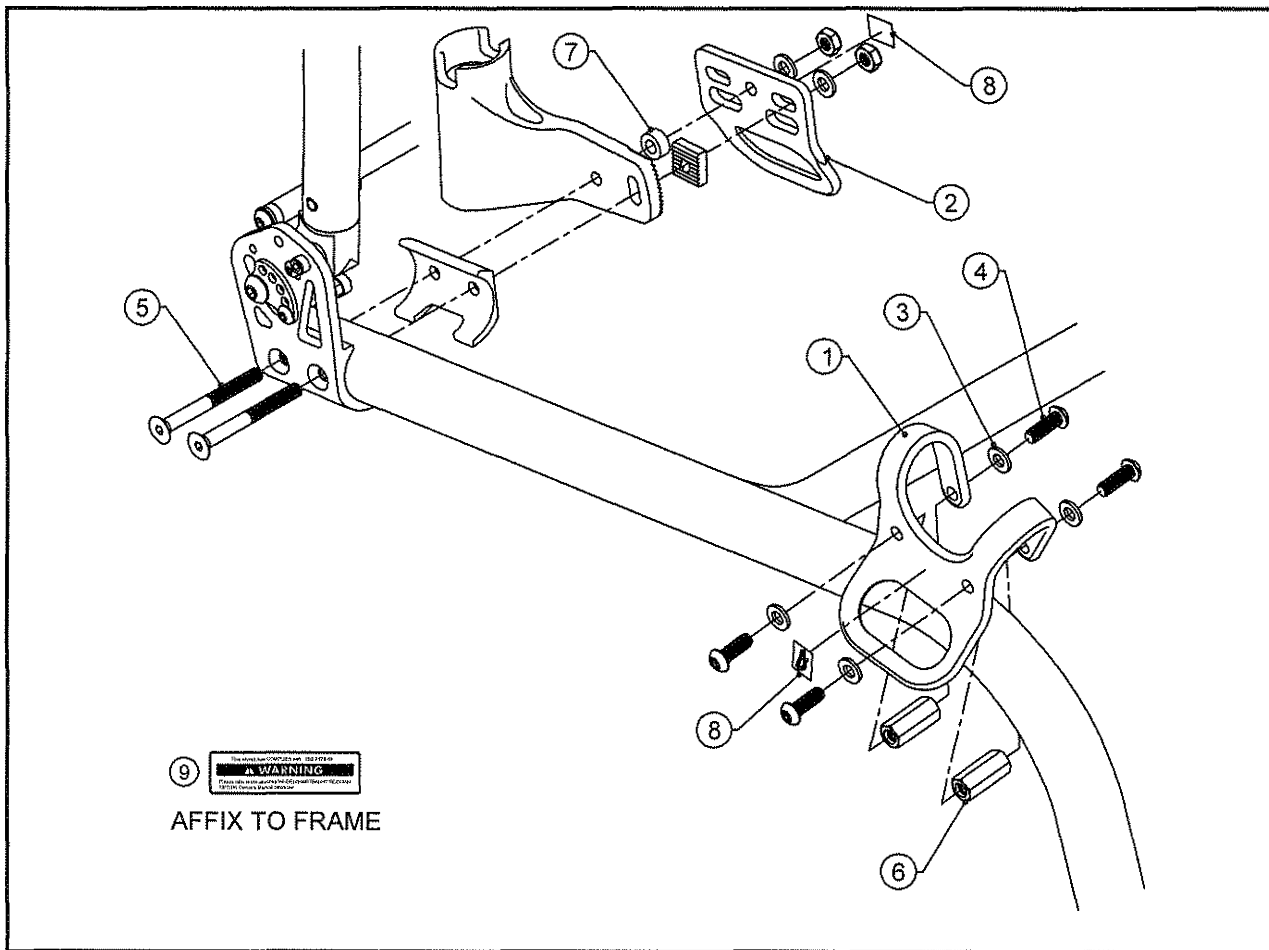

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP104971	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100646	1/4-20 X 3/4" BUTTON HEAD CAP SCREW, ALLOY STEEL, BLACK ZINC
5	4	HD100647	1/4-28 X 2-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	4	CP104972	COUPLING NUT 1/4-20 X 1-1/16"
7	2	HD100394	SPACER
8	4	CP103109	DECAL - TRANSPORT HOOK
9	1	CP105208	DECAL - ISO 7176-19 WARNING
1-9		AS111233	TRANSIT TIE-DOWN PACKAGE FOR USE W/ ARMRESTS

TRANSIT TIE-DOWN OPTION W/ TRANSIT SEATBELT BRACKET W/ ARMRESTS


TiLite™ DEPTH ADJUSTABLE AERO Z (SERIES 2), DEPTH ADJUSTABLE AERO Z TITANIUM (SERIES 2), DEPTH ADJUSTABLE ZRA (SERIES 2)

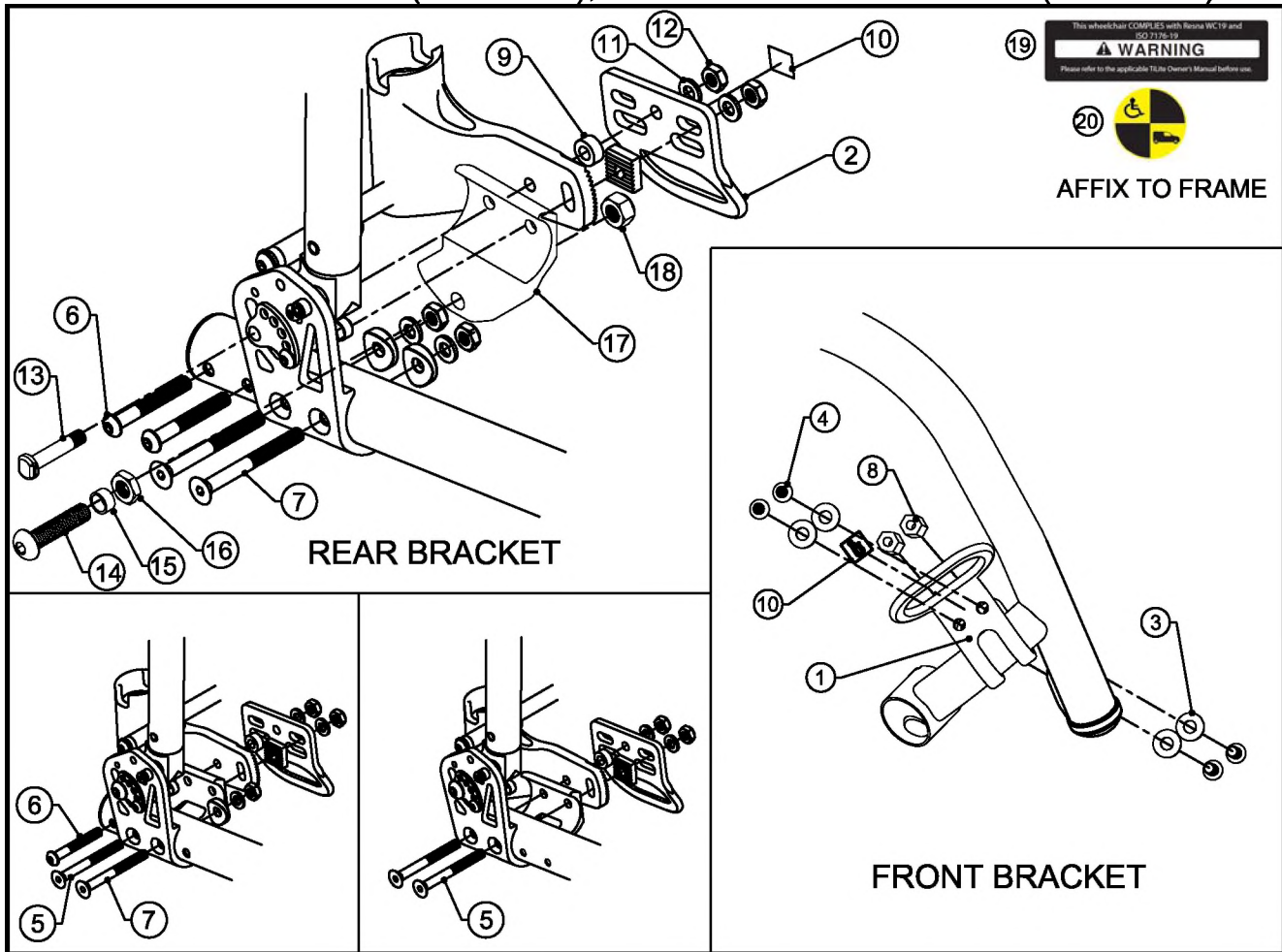

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	HD100730	1/4-28 X 2-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	4	HD100734	1/4-28 X 2-1/2" BHCS, BLACK ZINC TIEDOWN
7	4	HD100728	1/4-28 X 2" FLAT HEAD CAP SCREW, BLACK ZINC
8	4	CP108286	COUPLING NUT 1/4-20 X .690"
9	2	HD100394	SPACER
10	4	CP103109	DECAL - TRANSPORT HOOK
11	8	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
12	8	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
13	2	HD100732	STOP BOLT
14	2	HD100726	M8 X 35MM BHCS
15	2	HD100719	NYLON LAP BELT SPACER
16	2	HD100717	M8 THIN NUT, BLACK ZINC
17	2	CP107385	1.25" TRANSIT SEATBELT BRACKET
18	2	HD100718	M8 LOCK NUT, BLACK ZINC
19	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
20	2	RW100343	TRANSIT TIE-DOWN NOTIFICATION DECAL
1-4,8-12,19,20		AS117376	TRANSIT TIE-DOWN BRACKET ASSY WITH DEPTH ADJUSTABLE AND ARMREST
5-7,13-18		AS117590	1.25" SEATBELT BRACKET AND STOP BOLT DEPTH ADJ W/ ARMREST

TRANSIT TIE-DOWN OPTION W/ TRANSIT SEATBELT BRACKET


TiLite™ DEPTH ADJUSTABLE AERO Z (SERIES 2), DEPTH ADJUSTABLE AERO Z TITANIUM (SERIES 2), DEPTH ADJUSTABLE ZRA (SERIES 2)

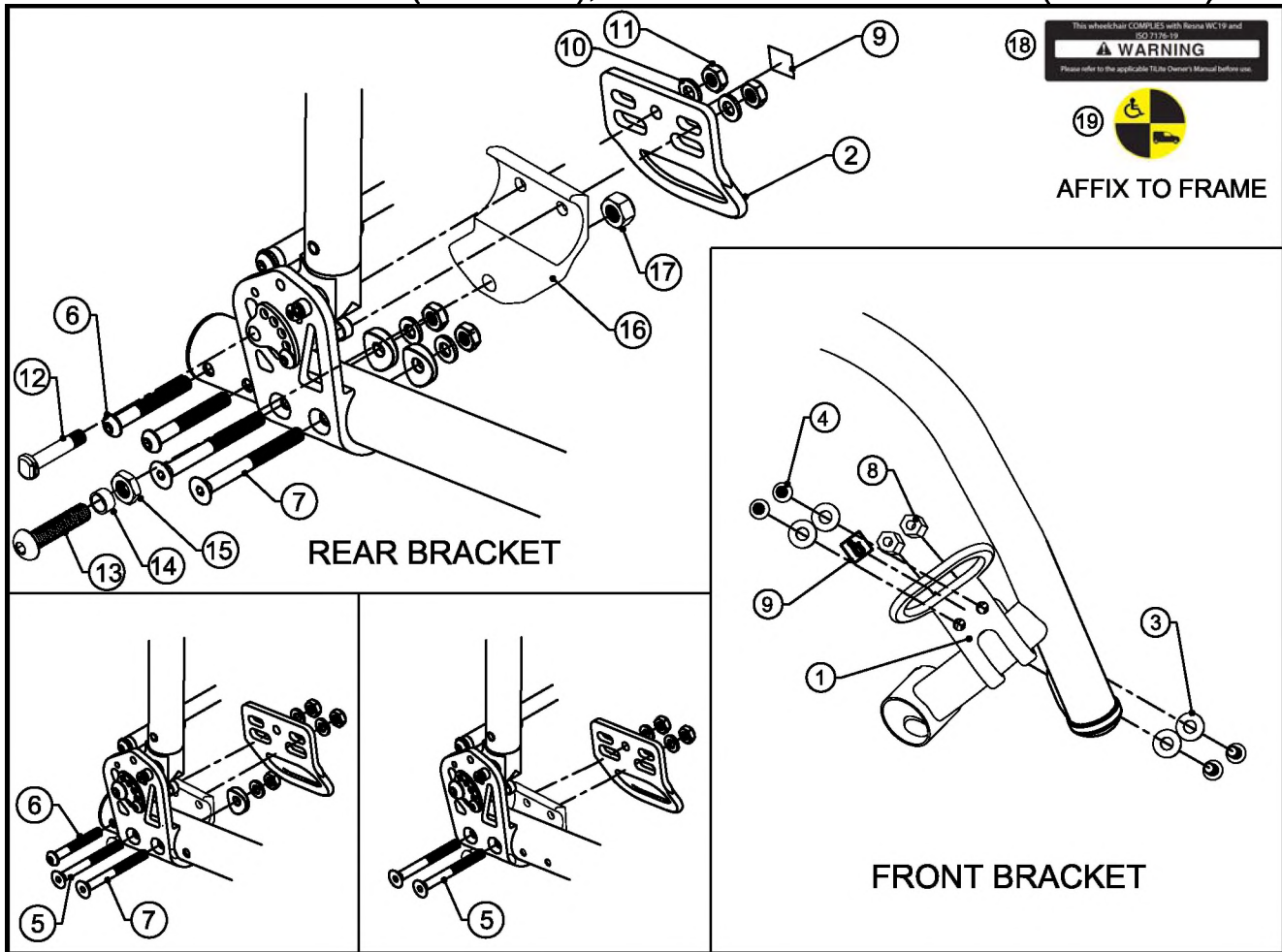

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	HD100727	1/4-28 X 2-1/4" FLAT HEAD CAP SCREW BLACK ZINC
6	4	HD100735	1/4-28 X 2" BHCS, BLACK ZINC TIEDOWN
7	4	HD100728	1/4-28 X 2" FLAT HEAD CAP SCREW BLACK ZINC
8	4	CP108286	COUPLING NUT 1/4-20 X .690"
9	4	CP103109	DECAL - TRANSPORT HOOK
10	8	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
11	8	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
12	2	HD100732	STOP BOLT
13	2	HD100726	M8 X 35MM BHCS
14	2	HD100719	NYLON LAP BELT SPACER
15	2	HD100717	M8 THIN NUT, BLACK ZINC
16	2	CP107385	1.25" TRANSIT SEATBELT BRACKET
17	2	HD100718	M8 LOCK NUT, BLACK ZINC
18	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
19	2	RW100343	TRANSIT TIE-DOWN NOTIFICATION DECAL
1-4,8-11,18,19		AS117456	TRANSIT TIE-DOWN BRACKET ASSEMBLY WITH DEPTH ADJUSTABLE
5-7,12-17		AS117589	1.25" SEATBELT BRACKET AND STOP BOLT DEPTH ADJ

TRANSIT TIE-DOWN OPTION FOR USE W/ ARMRESTS


TiLite™ DEPTH ADJUSTABLE ZRA (SERIES 2),
DEPTH ADJUSTABLE AERO Z (TITANIUM FRAME)

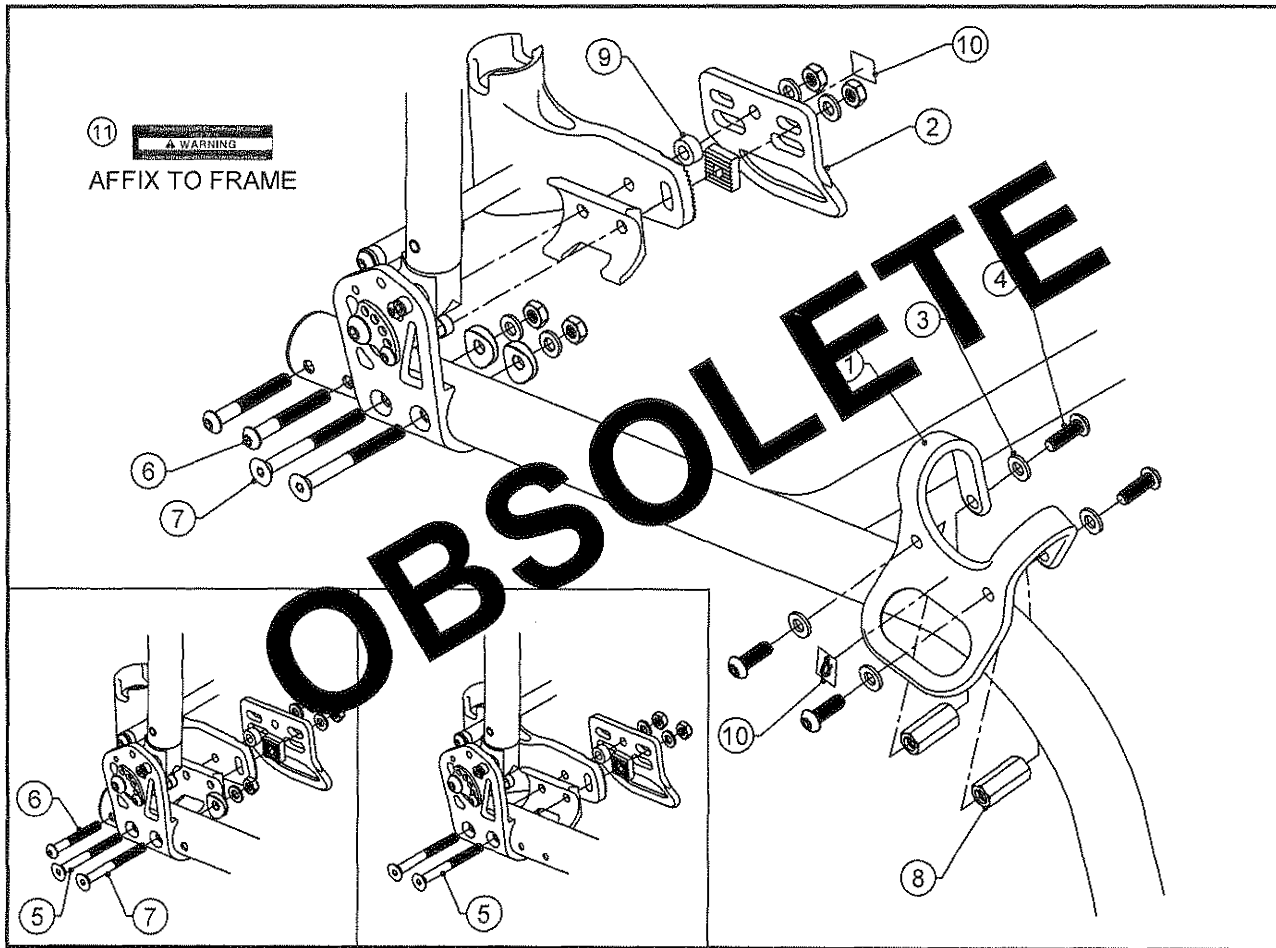

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP104971	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100646	1/4-20 X 3/4" BUTTON HEAD CAP SCREW, ALLOY STEEL, BLACK ZINC
5	4	HD100647	1/4-28 X 2-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	4	HD100655	1/4-28 X 2-1/2" BUTTON HEAD CAP SCREW, BLACK ZINC
7	4	HD100605	1/4-28 X 2" FLAT HEAD CAP SCREW BLACK ZINC
8	4	CP104972	COUPLING NUT 1/4-20 X 1-1/16"
9	2	HD100394	SPACER
10	4	CP103109	DECAL - TRANSPORT HOOK
11	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
1-11		AS111339	DEPTH ADJ TRANSIT TIE-DOWN PACKAGE FOR USE W/ ARMRESTS

TRANSIT TIE-DOWN OPTION


TiLite™ TWIST

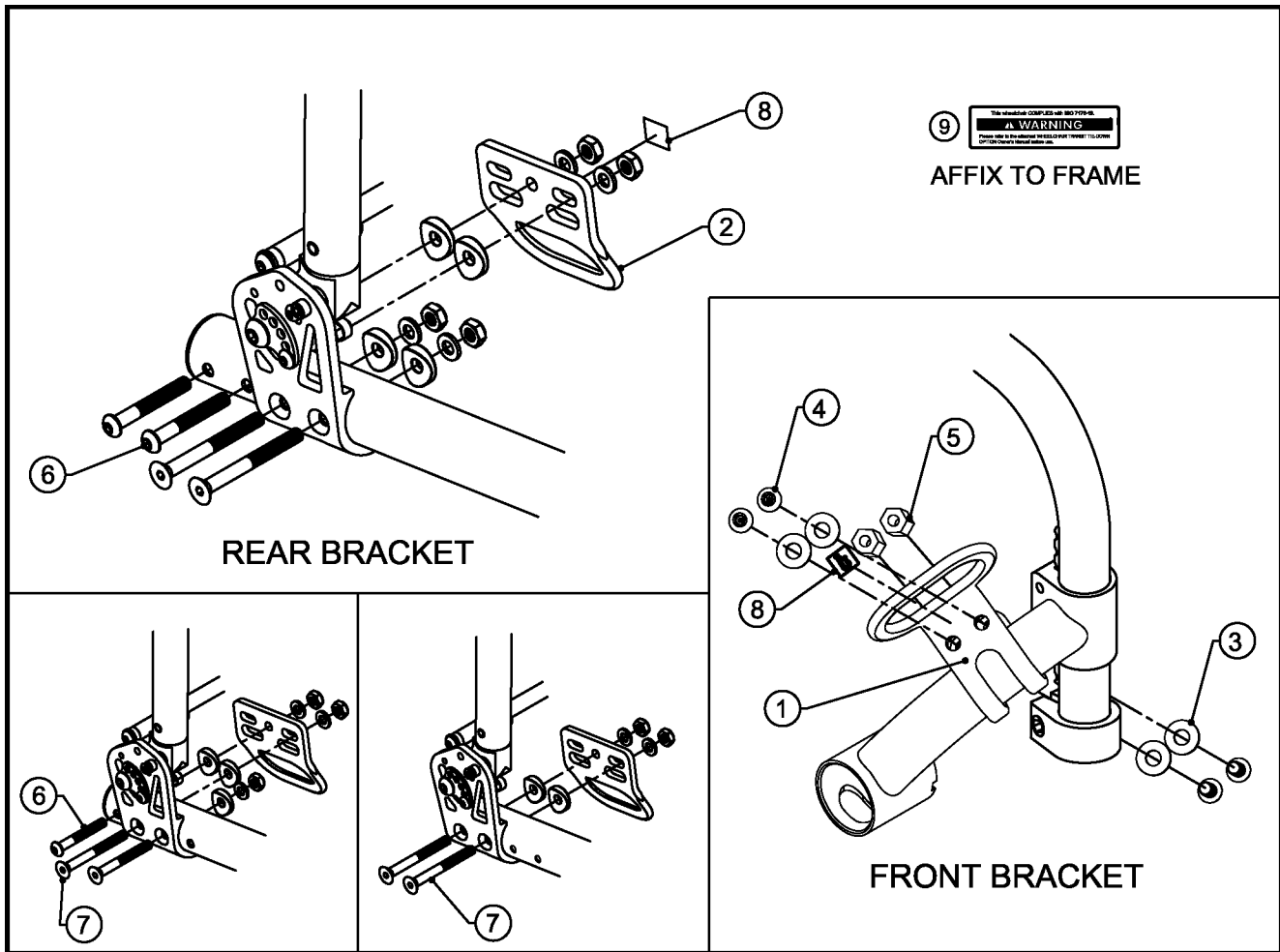

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2" BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	CP108286	COUPLING NUT 1/4-20 X .690"
6	4	HD100485	1/4-28 X 1-5/8" BUTTON HEAD CAP SCREW, BLACK ZINC
7	4	HD100605	1/4-28 X 2" FLAT HEAD CAP SCREW, BLACK ZINC
8	4	CP103109	DECAL - TRANSPORT HOOK
9	1	CP105208	DECAL - ISO 7176-19 WARNING
1-9		AS111877	TRANSIT TIE-DOWN PACKAGE

TRANSPORT BRACKET OPTION

TiLite™ TWIST

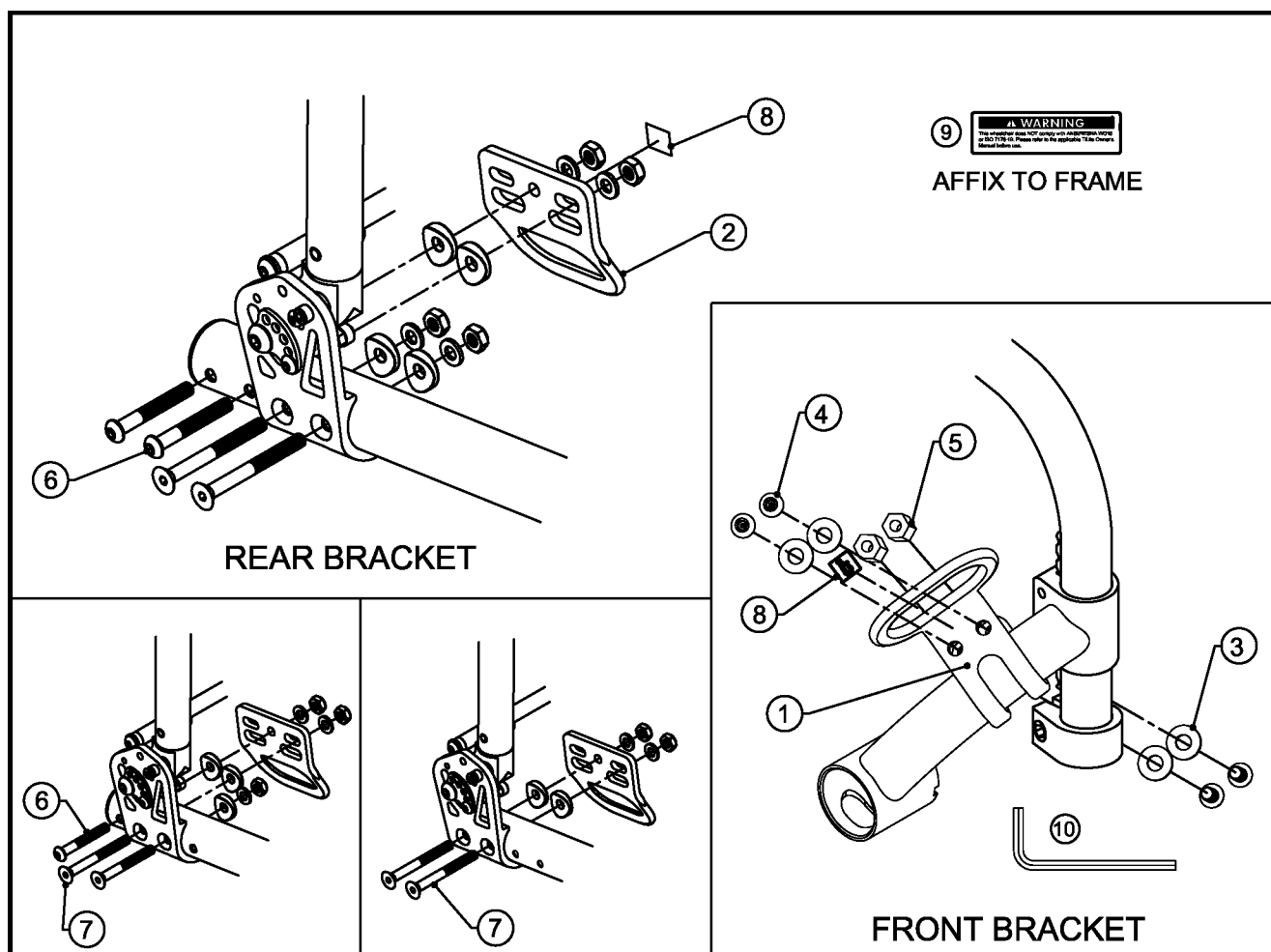


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2" BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	CP108286	COUPLING NUT 1/4-20 X .690"
6	4	HD100485	1/4-28 X 1-5/8" BUTTON HEAD CAP SCREW, BLACK ZINC
7	4	HD100605	1/4-28 X 2" FLAT HEAD CAP SCREW, BLACK ZINC
8	4	CP103109	DECAL - TRANSPORT HOOK
9	1	CP105025	DECAL - TRANSPORT BRACKET
10	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*11	1	DC100890	TRANSPORT BRACKET FIELD ASSEMBLY GUIDE
*12	1	DC100895	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*13	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-9		AS115942	TRANSPORT BRACKET PACKAGE
1-13		AS115941-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT

* NOT SHOWN

TRANSIT TIE-DOWN OPTION FOR USE W/ ARMRESTS


TiLite™ TWIST

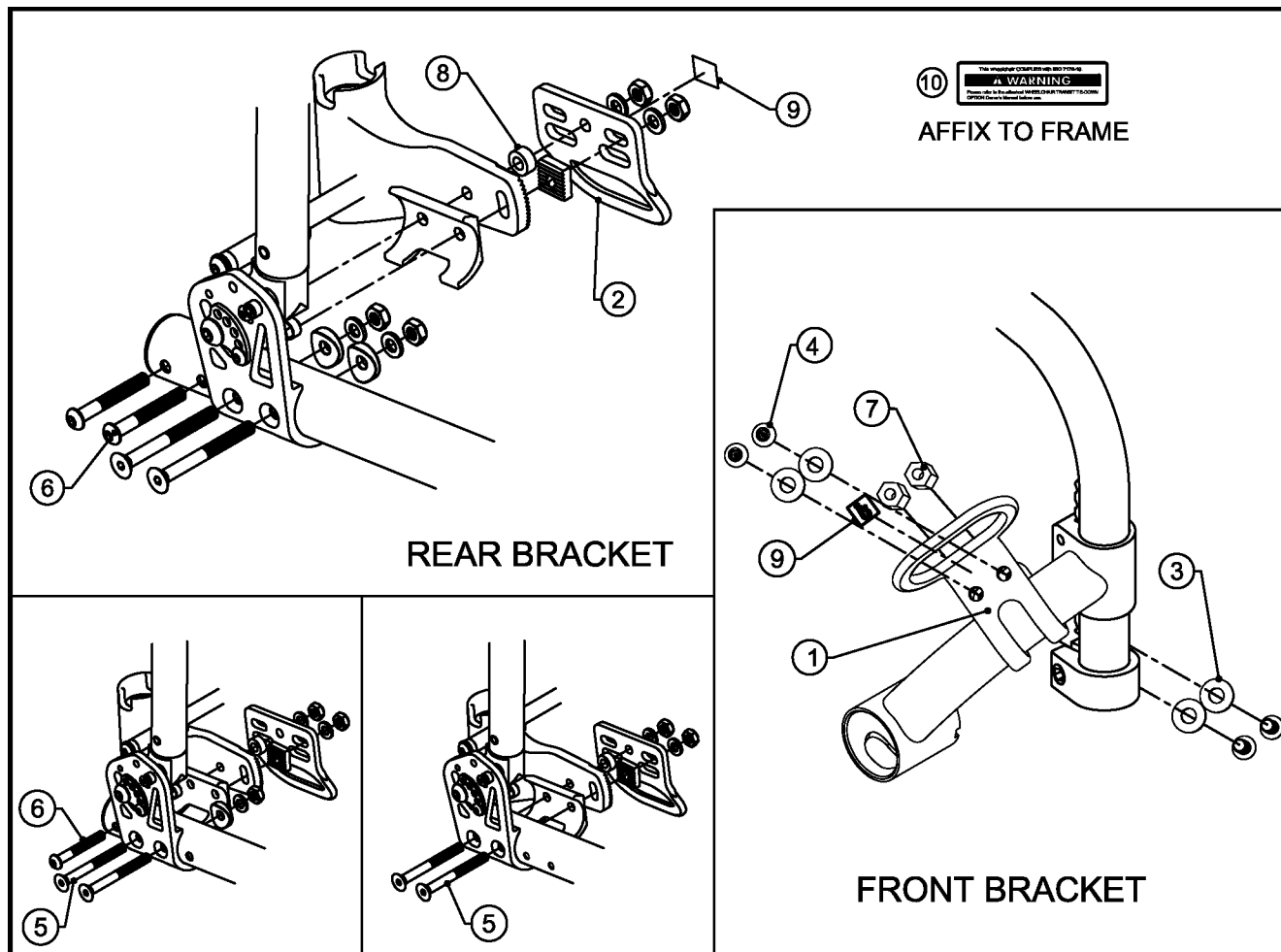

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2" BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	HD100604	1/4-28 X 2-1/2" FLAT HEAD CAP SCREW BLACK ZINC
6	4	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
7	4	CP108286	COUPLING NUT 1/4-20 X .690"
8	2	HD100394	SPACER
9	4	CP103109	DECAL - TRANSPORT HOOK
10	1	CP105208	DECAL - ISO 7176-19 WARNING
1-10		AS111878	TRANSIT TIE-DOWN PACKAGE FOR USE W/ ARMRESTS

TRANSPORT BRACKET OPTION FOR USE W/ ARMRESTS

TiLite™ TWIST

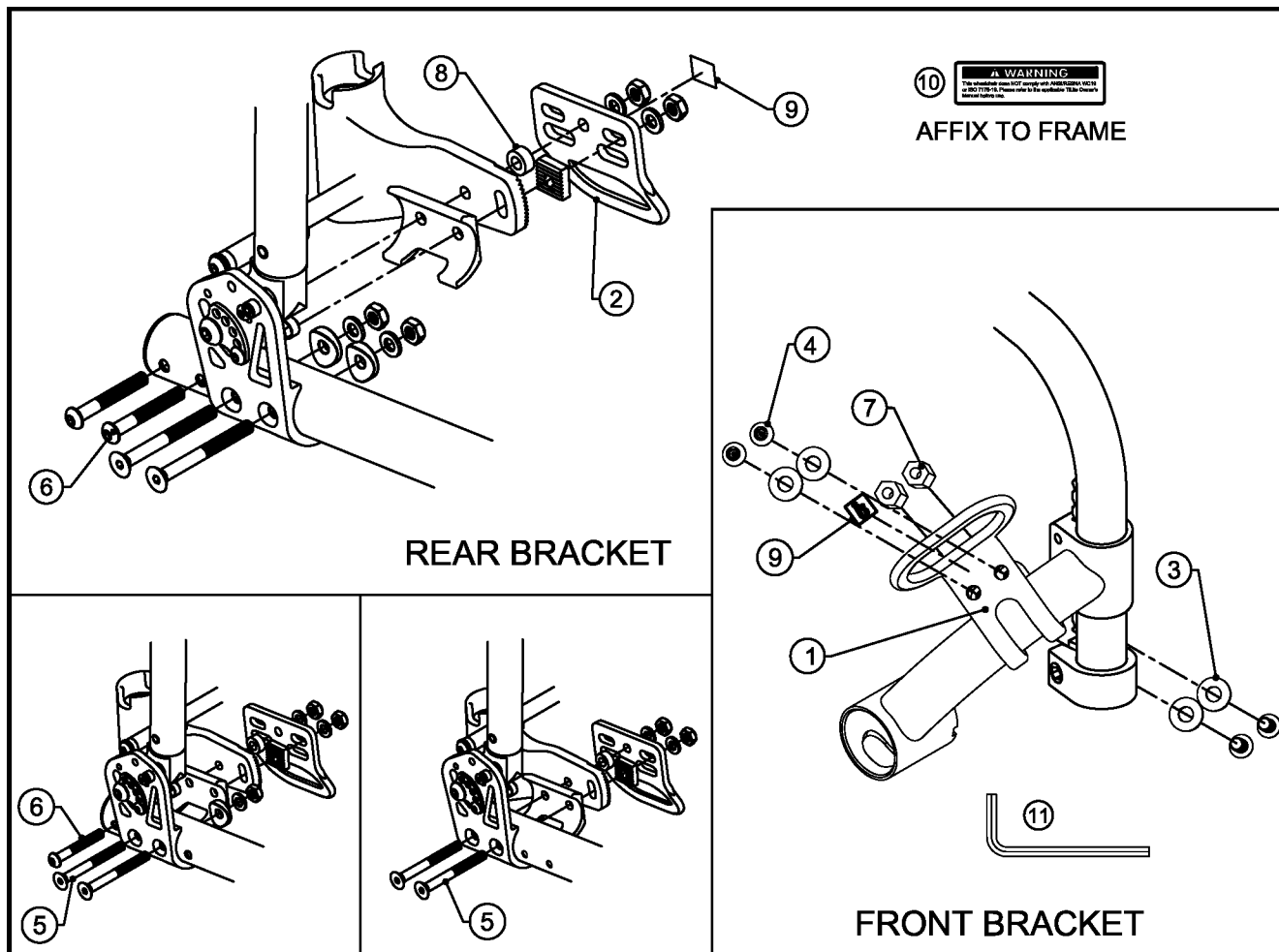


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2" BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	HD100604	1/4-28 X 2-1/2" FLAT HEAD CAP SCREW BLACK ZINC
6	4	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
7	4	CP108286	COUPLING NUT 1/4-20 X .690"
8	2	HD100394	SPACER
9	4	CP103109	DECAL - TRANSPORT HOOK
10	1	CP105025	DECAL - TRANSPORT BRACKET
11	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*12	1	DC100890	TRANSPORT BRACKET FIELD ASSEMBLY GUIDE
*13	1	DC100895	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*14	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-10		AS115943	TRANSPORT BRACKET PACKAGE FOR USE W/ ARMRESTS
1-14		AS118850-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT FOR USE W/ ARMRESTS

* NOT SHOWN

TRANSIT TIE-DOWN OPTION

TiLite™ Aero X

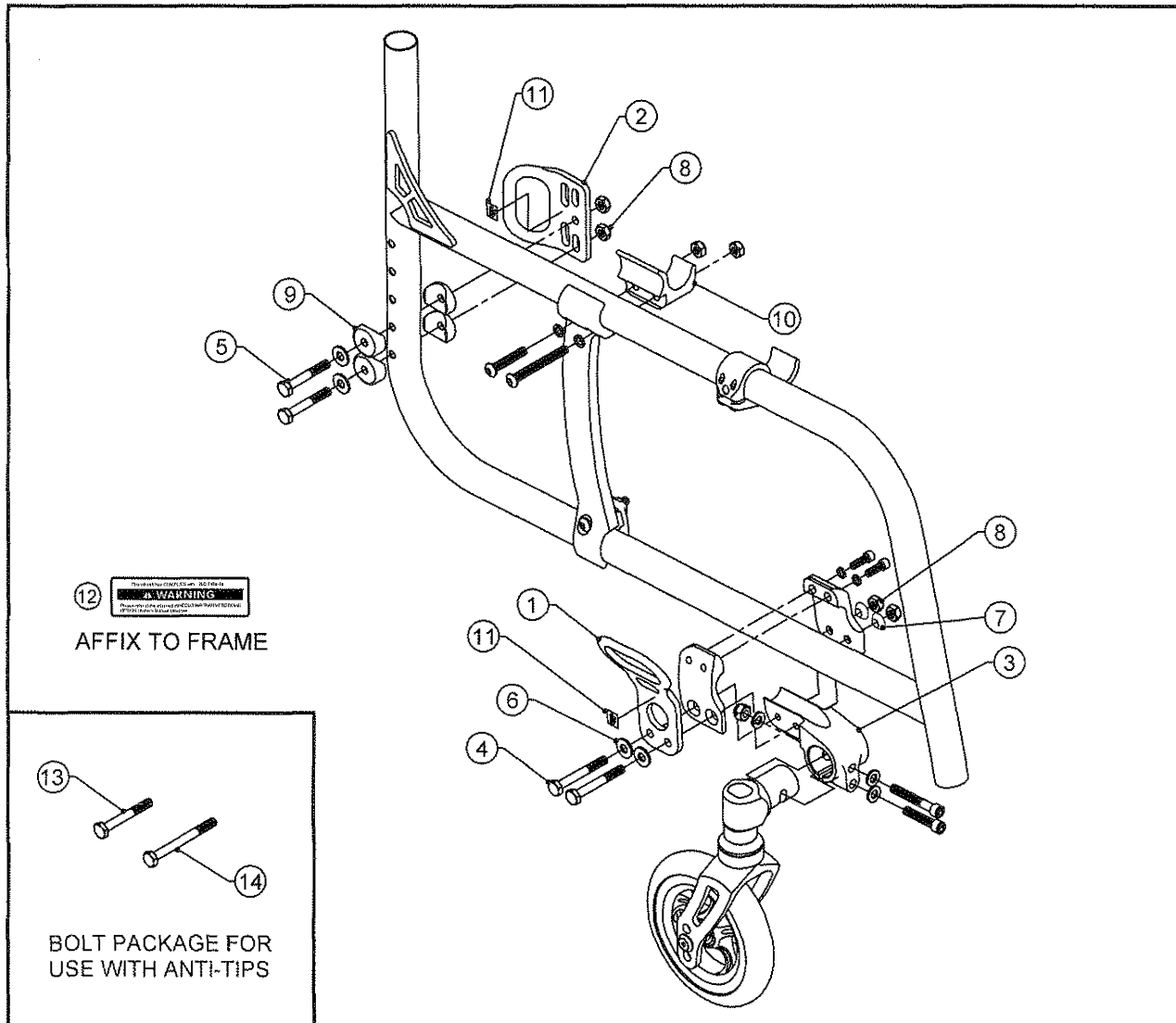

ITEM	QTY	PART NO.	DESCRIPTION
1	2	CP104973	EXTENDED FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	2	CP104974	HD CASTER MOUNT
4	4	HD100690	1/4-28 X 2" X 3/4" THD HEX CAP SCREW, GRADE 5, ZINC PLATE
5	4	HD100691	1/4-28 X 1-3/4" X 3/4" THD HEX CAP SCREW, GRADE 5, ZINC PLATE
6	8	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
7	4	CP104721	1/4" FLATHEAD SPACER
8	8	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
9	8	HD100014	COVERED WASHER 1" OD X 1/4" ID X 1/8" THK
10	2	CP105808	ALUMINUM AXLE PLATE SADDLE
11	4	CP103109	DECAL - TRANSPORT HOOK
12	1	CP105208	DECAL - ISO 7176-19 WARNING
13	4	HD100690	1/4-28 X 2" X 3/4" THD HEX CAP SCREW, GRADE 5, ZINC PLATE
14	4	HD100457	1/4-28 X 2-3/4" HEX CAP SCREW, GRADE 8, ZINC PLATE
1-12		AS111947	TRANSIT TIE DOWN PACKAGE
13-14		AS111951	TRANSIT TIE DOWN BOLT PACK


TRANSIT TIE DOWN OPTION INCLUDES:

TRANSPORT BRACKETS (SHOWN)
 REINFORCED FRAME (SEE PAGES 2-13.5.5 AND 2-13.5.6)
 FOLDING STABILIZER BAR (SEE PAGES 10-18.3 AND 10-18.5)
 HEAVY DUTY CASTER MOUNTS (SEE PAGE 8-1.7)
 REINFORCED CROSSTUBES (SEE PAGE 2-17.9)



TRANSPORT BRACKET OPTION

TiLite™ 2GX, AERO X

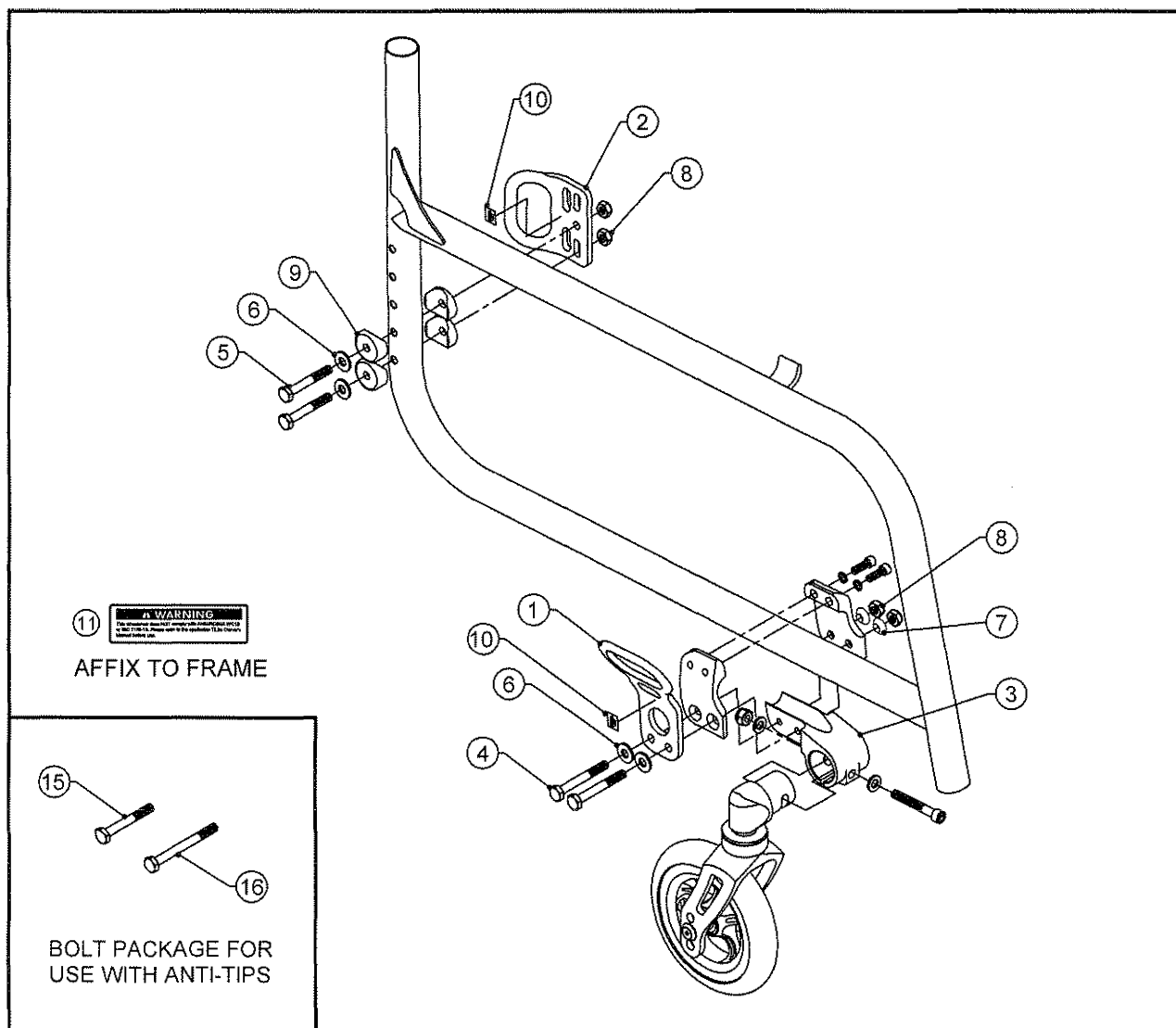

ITEM	QTY	PART NO.	DESCRIPTION
1	2	CP104973	EXTENDED FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	2	CP106262	CASTER MOUNT
4	4	HD100690	1/4-28 X 2" X 3/4" THD HEX CAP SCREW, GRADE 5, ZINC PLATE
5	4	HD100691	1/4-28 X 1-3/4" X 3/4" THD HEX CAP SCREW, GRADE 5, ZINC PLATE
6	8	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
7	4	CP104721	1/4" FLATHEAD SPACER
8	8	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
9	8	HD100014	COVERED WASHER 1" OD X 1/4" ID X 1/8" THK
10	4	CP103109	DECAL - TRANSPORT HOOK
11	1	CP105025	DECAL - TRANSPORT BRACKET
*12	1	DC100892	TRANSPORT BRACKET FIELD ASSEMBLY GUIDE
*13	1	DC100893	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*14	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
15	4	HD100690	1/4-28 X 2" X 3/4" THD HEX CAP SCREW, GRADE 5, ZINC PLATE
16	4	HD100457	1/4-28 X 2-3/4" HEX CAP SCREW, GRADE 8, ZINC PLATE
1-11		AS112040	TRANSPORT BRACKET PACKAGE
1-14		AS115940-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT
15-16		AS111951	TRANSPORT BRACKET BOLT PACK


* NOT SHOWN



Parts Manual

TRANSPORT BRACKET OPTION

TiLite™ ZRA (SERIES 2), TiLite™ AERO Z (SERIES 2)

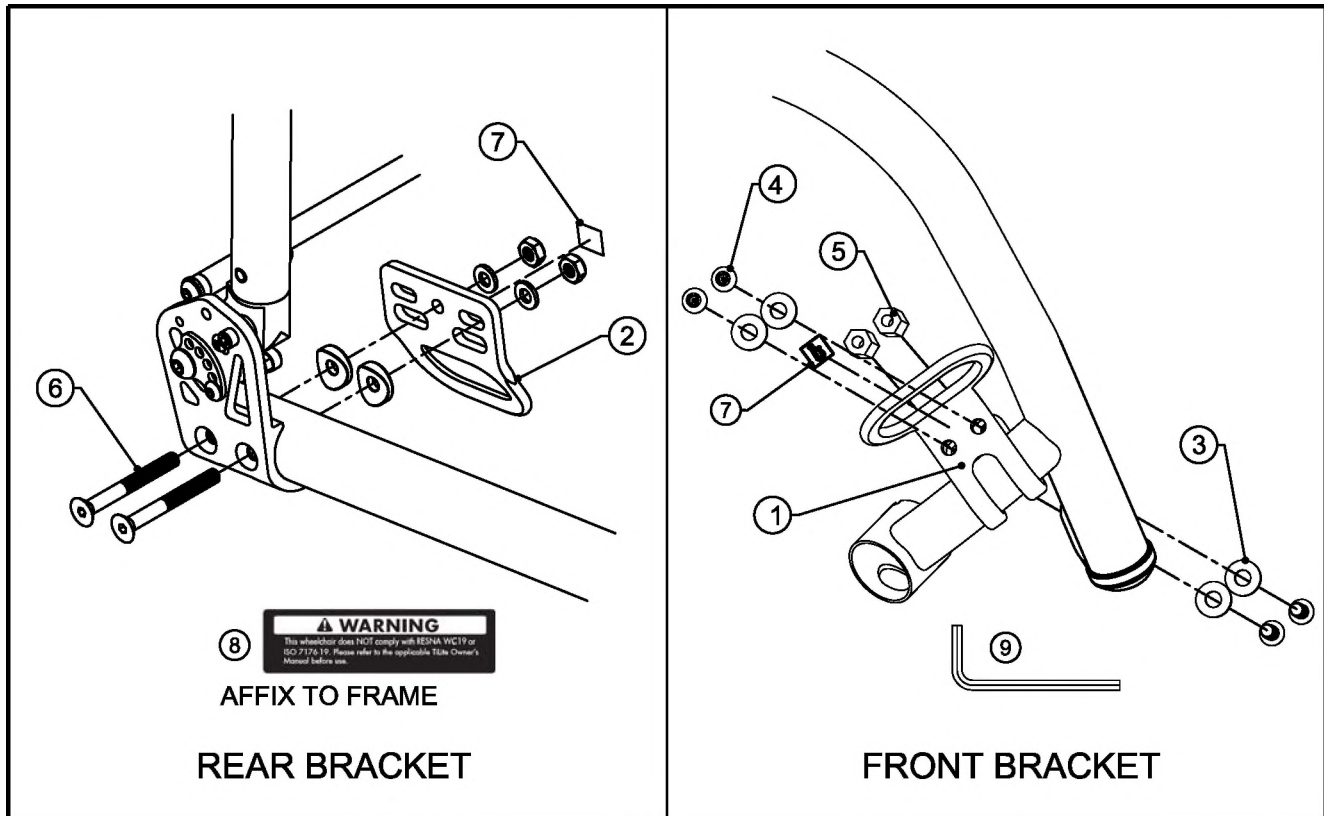


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	CP108286	COUPLING NUT 1/4-20 X .690"
6	4	HD100606	1/4-28 X 2-1/4" FLAT HEAD CAP SCREW BLACK ZINC
7	4	CP103109	DECAL - TRANSPORT HOOK
8	1	RW100344	TRANSPORT BRACKET DECAL
9	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*10	1	DC100873	TRANSPORT BRACKET FIELD ASSEMBLY GUIDE
*11	1	DC100894	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*12	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-8		AS115935	TRANSPORT BRACKET PACKAGE
1-12		AS115920-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT

* NOT SHOWN

TRANSPORT BRACKET OPTION FOR USE W/ ARMRESTS

TiLite™ ZRA (SERIES 2), TiLite™ AERO Z (SERIES 2)

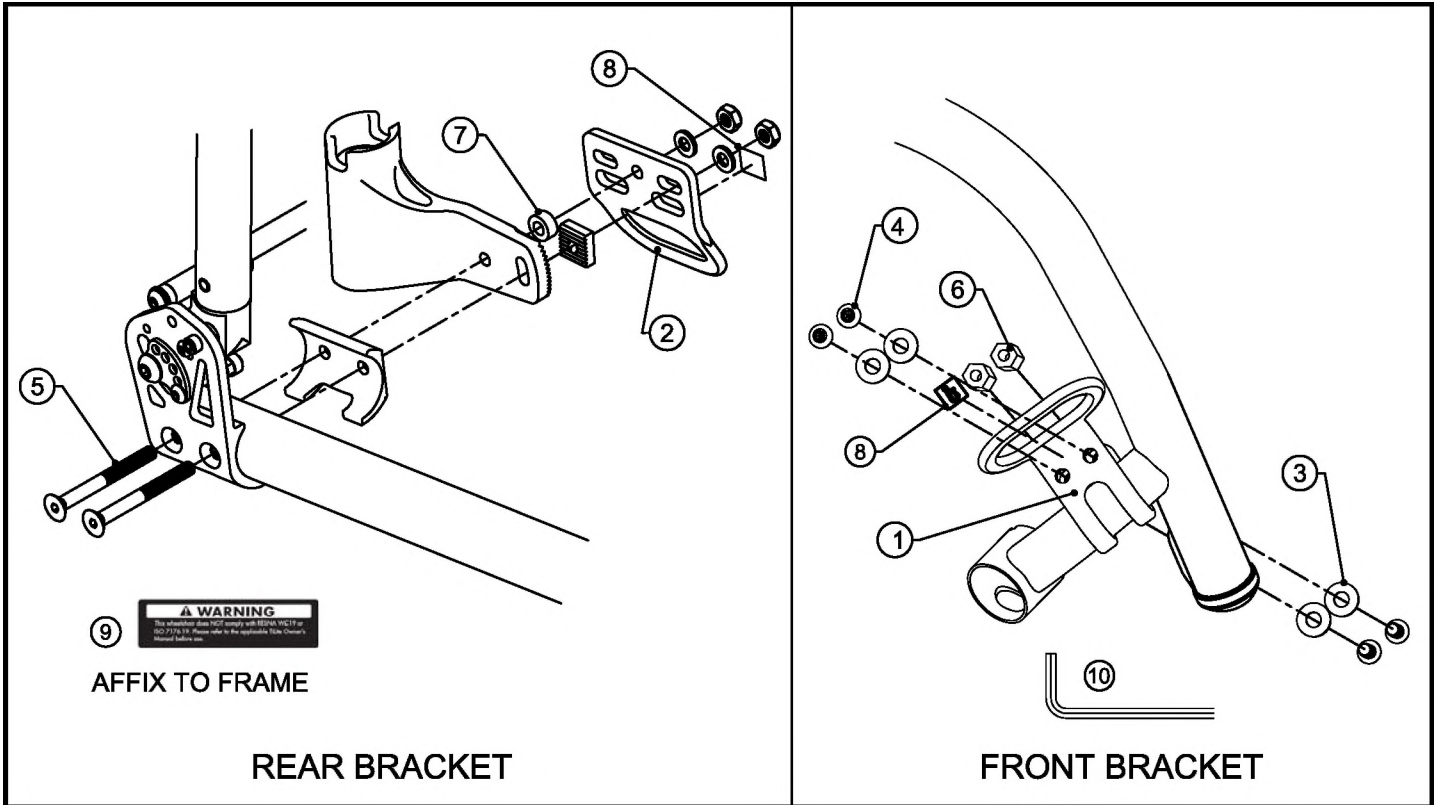


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	HD100647	1/4-28 X 2-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	4	CP108286	COUPLING NUT 1/4-20
7	2	HD100394	SPACER
8	4	CP103109	DECAL - TRANSPORT HOOK
9	1	RW100344	TRANSPORT BRACKET DECAL
10	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*11	1	DC100873	TRANSPORT BRACKET FIELD ASSEMBLY GUIDE
*12	1	DC100894	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*13	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-9		AS115908	TRANSPORT BRACKET PACKAGE FOR USE W/ ARMRESTS
1-13		AS117587-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT W/ ARMREST

*NOT SHOWN

TRANSPORT BRACKET OPTION

TiLite™ DEPTH ADJUSTABLE ZRA (SERIES 2),
TiLite™ DEPTH ADJUSTABLE AERO Z (SERIES 2)

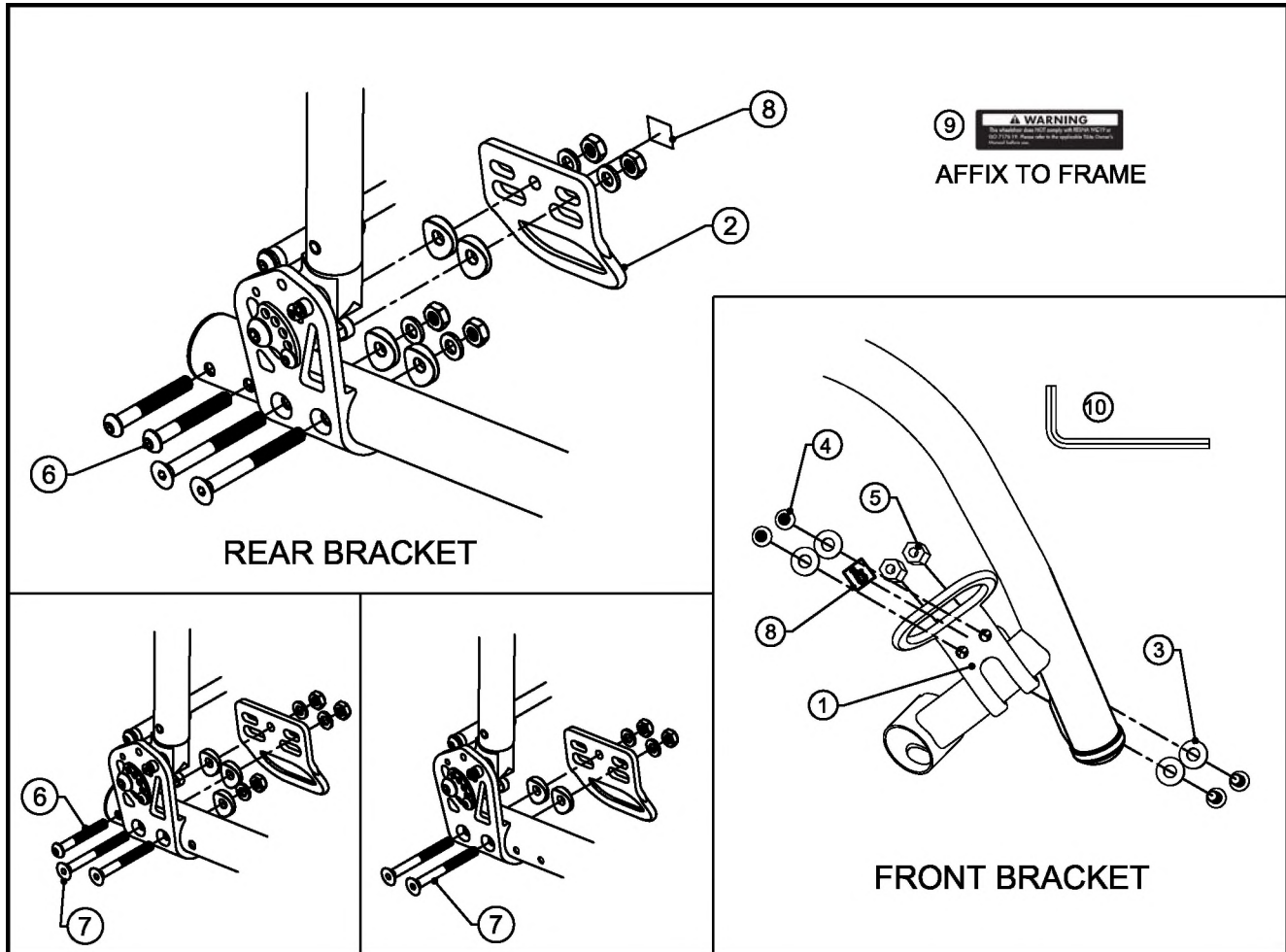


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	CP108286	COUPLING NUT 1/4-20 X .690"
6	4	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
7	4	HD100606	1/4-28 X 2-1/4" FLAT HEAD CAP SCREW BLACK ZINC
8	4	CP103109	DECAL - TRANSPORT HOOK
9	1	RW100344	TRANSPORT BRACKET DECAL
10	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*11	1	DC100916	FIELD ASSEMBLY GUIDE: INSTALLING THE TRANSPORT BRACKET OPTION ON THE TILITE AERO Z / ZRA depth adjus
*12	1	DC100894	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*13	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-9		AS115936	DEPTH ADJ TRANSPORT BRACKET PACKAGE
1-13		AS117585-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT DEPTH ADJ ZRA, AERO Z

* NOT SHOWN

TRANSPORT BRACKET OPTION FOR USE W/ ARMRESTS

TiLite™ DEPTH ADJUSTABLE ZRA (SERIES 2),
TiLite™ DEPTH ADJUSTABLE AERO Z (SERIES 2)

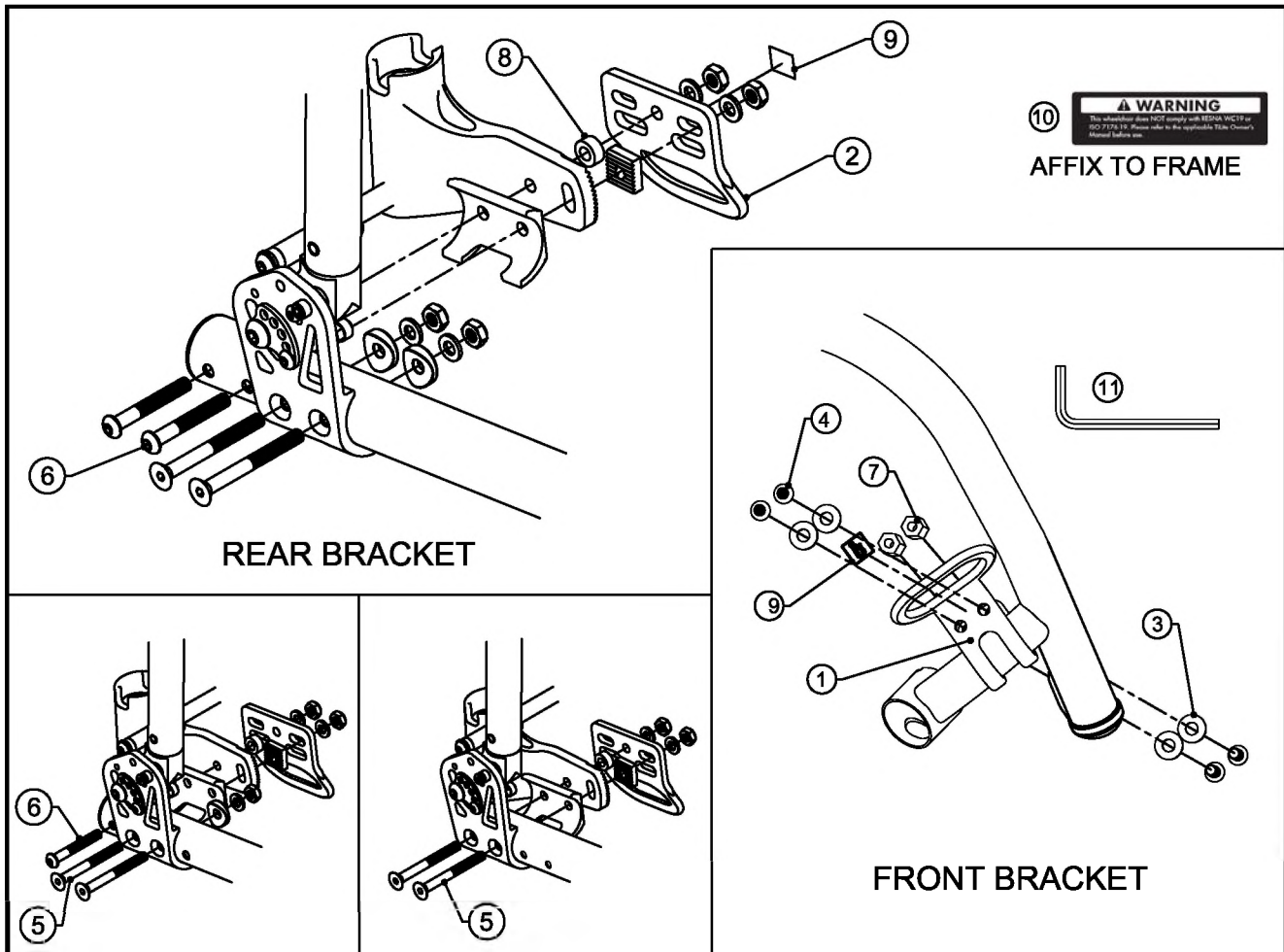


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP106960	FRONT TRANSPORT BRACKET
2	2	CP102513	REAR TRANSPORT BRACKET
3	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
4	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
5	4	HD100647	1/4-28 X 2-3/4" FLAT HEAD CAP SCREW, BLACK ZINC
6	4	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
7	4	CP108286	COUPLING NUT 1/4-20 X .690"
8	2	HD100394	SPACER
9	4	CP103109	DECAL - TRANSPORT HOOK
10	1	RW100344	TRANSPORT BRACKET DECAL
11	1	HD100700	5/32" TAMPER-PROOF HEX L-KEY, BLACK
*12	1	DC100916	FIELD ASSEMBLY GUIDE: INSTALLING THE TRANSPORT BRACKET OPTION ON THE TILITE AERO Z / ZRA DEPTH ADJUSTABLE
*13	1	DC100894	TRANSPORT BRACKET OWNER'S MANUAL SUPPLEMENT
*14	1	DC100882	TRANSPORT BRACKET OPTION PRE-INSTALLATION CHECKLIST
1-10		AS115909	DEPTH ADJ TRANSPORT BRACKET PACKAGE FOR USE W/ ARMRESTS
1-14		AS117586-I	TRANSPORT BRACKET OPTION - FIELD INSTALL KIT

* NOT SHOWN

TRANSIT TIEDOWN OPTION W/ TRANSIT SEATBELT BRACKET

ZRA(SERIES 2), AERO Z(SERIES 2), AERO Z TITANIUM(SERIES 2)

REAR BRACKET

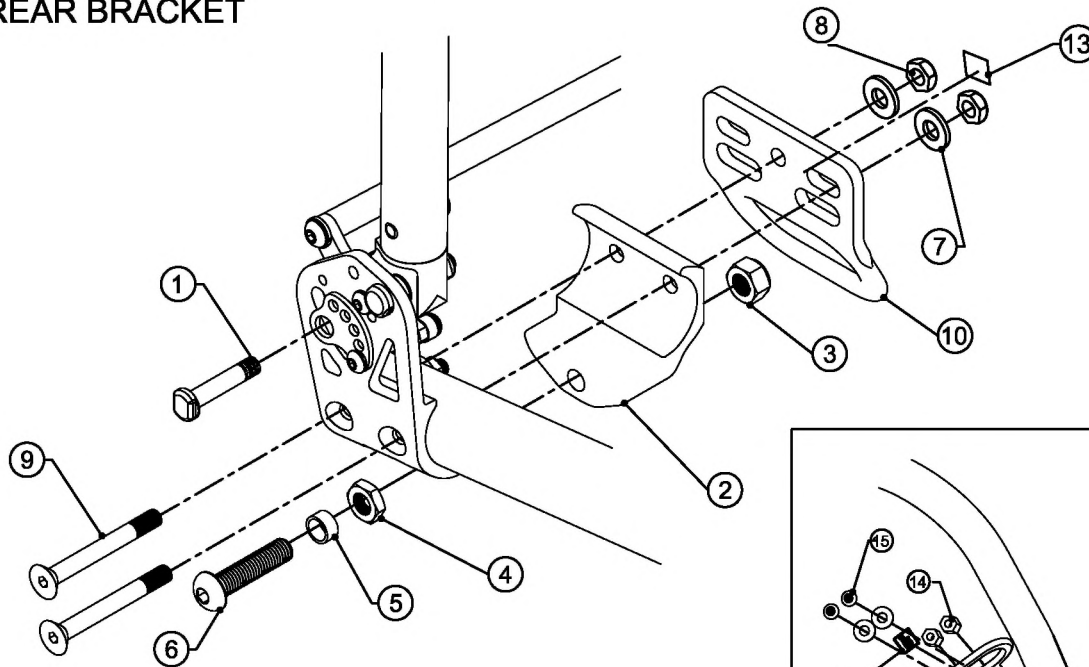


17



16

AFFIX TO FRAME

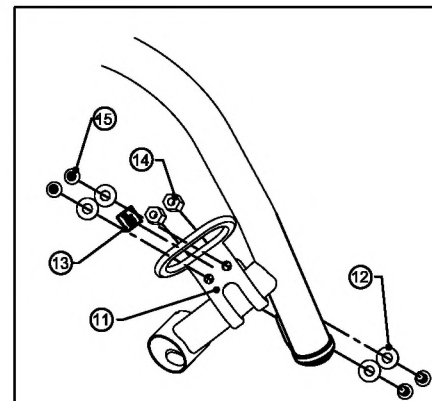


FRONT BRACKET

ITEM	QTY	PART NO.	DESCRIPTION
1	2	HD100732	STOP BOLT
2	2	CP107385	1.25" TRANSIT SEATBELT BRACKET
3	2	HD100718	M8 LOCK NUT, BLACK ZINC
4	2	HD100717	M8 THIN NUT, BLACK ZINC
5	2	HD100719	NYLON LAP BELT SPACER
6	2	HD100726	M8 X 35MM BHCS
7	4	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
8	4	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
9	4	HD100727	1/4-28 X 2-1/4" FLAT HEAD CAP SCREW BLACK ZINC
10	2	CP102513	TIE DOWN ANCHOR MOUNT
11	2	CP106960	TRANSPORT BRACKET
12	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
13	4	CP103109	DECAL - TRANSPORT HOOK
14	4	CP108286	COUPLING NUT 1/4-20 X .690"
15	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
16	2	RW100343	TRANSIT TIE-DOWN NOTIFICATION DECAL
17	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
7,8,10-17		AS117588	TRANSIT TIE-DOWN BRACKET ASSEMBLY STANDARD
1-6,9		AS117377	1.25" SEATBELT BRACKET AND STOP BOLT

TRANSIT TIEDOWN OPTION W/ TRANSIT SEATBELT BRACKET W/ ARMRESTS


ZRA(SERIES 2), AERO Z(SERIES 2), AERO Z TITANIUM(SERIES 2)

REAR BRACKET

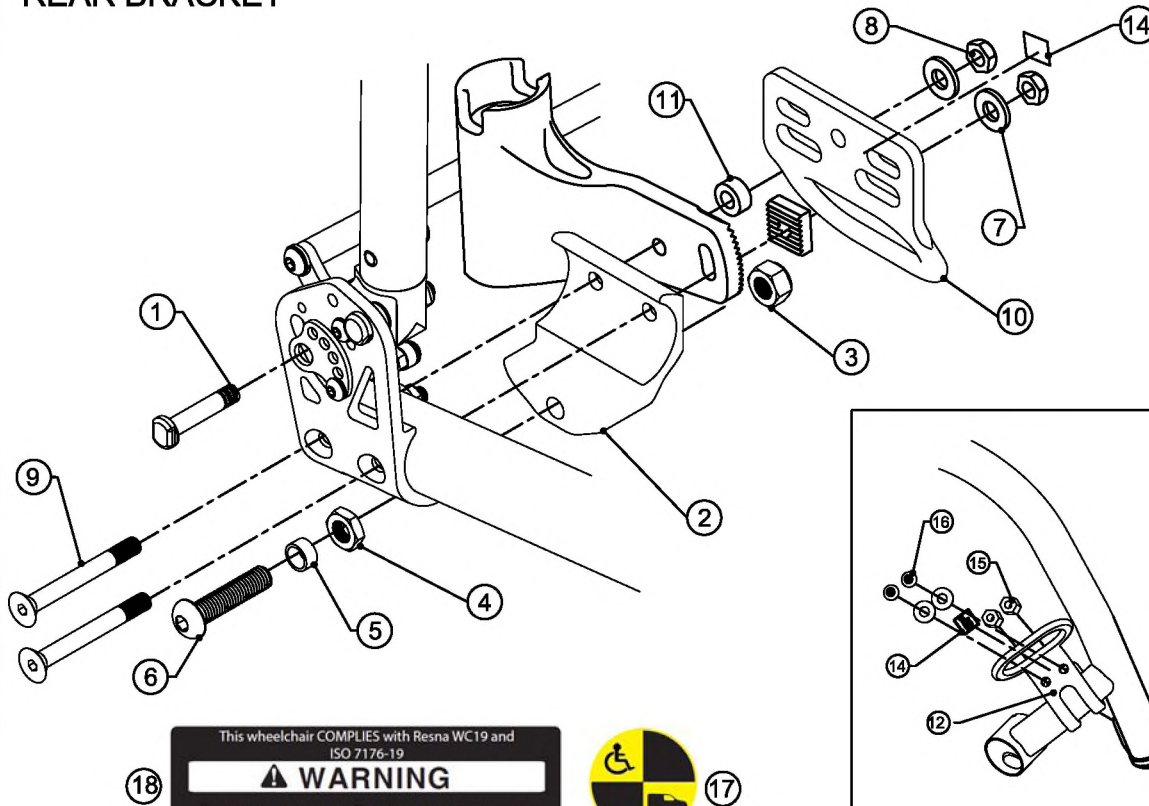

AFFIX TO FRAME


FRONT BRACKET

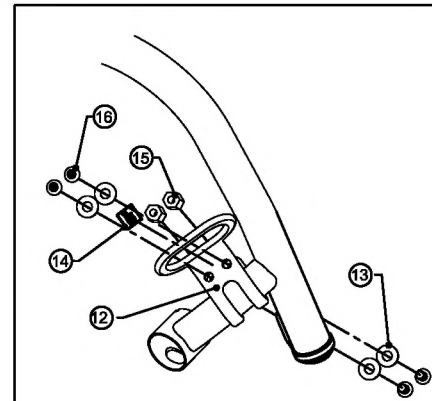

ITEM	QTY	PART NO.	DESCRIPTION
1	2	HD100732	STOP BOLT
2	2	CP107385	1.25" TRANSIT SEATBELT BRACKET
3	2	HD100718	M8 LOCK NUT, BLACK ZINC
4	2	HD100717	M8 THIN NUT, BLACK ZINC
5	2	HD100719	NYLON LAP BELT SPACER
6	2	HD100726	M8 X 35MM BHCS
7	4	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
8	4	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
9	4	HD100730	1/4-28 X 2-3/4" FLAT HEAD CAP SCREW BLACK ZINC
10	2	CP102513	TIE DOWN ANCHOR MOUNT
11	2	HD100394	SPACER
12	2	CP106960	TRANSPORT BRACKET
13	8	HD100618	1/4" LOCK WASHER, BLACK ZINC
14	4	CP103109	DECAL - TRANSPORT HOOK
15	4	CP108286	COUPLING NUT 1/4-20 X .690"
16	8	HD100748	1/4-20 X 1/2 BUTTON HEAD CAP SCREW TAMPER-PROOF, BLACK
17	2	RW100343	TRANSIT TIE-DOWN NOTIFICATION DECAL
18	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
7,8,10-18		AS117455	TRANSIT TIE-DOWN BRACKET ASSEMBLY WITH ARMREST
1-6,9		AS117584	1.25" SEATBELT BRACKET AND STOP BOLT W/ ARMREST

TRANSIT AND TRANSPORT BRACKET OPTIONS


TiLite™ 2GX (SERIES 2), AERO X (SERIES 2)

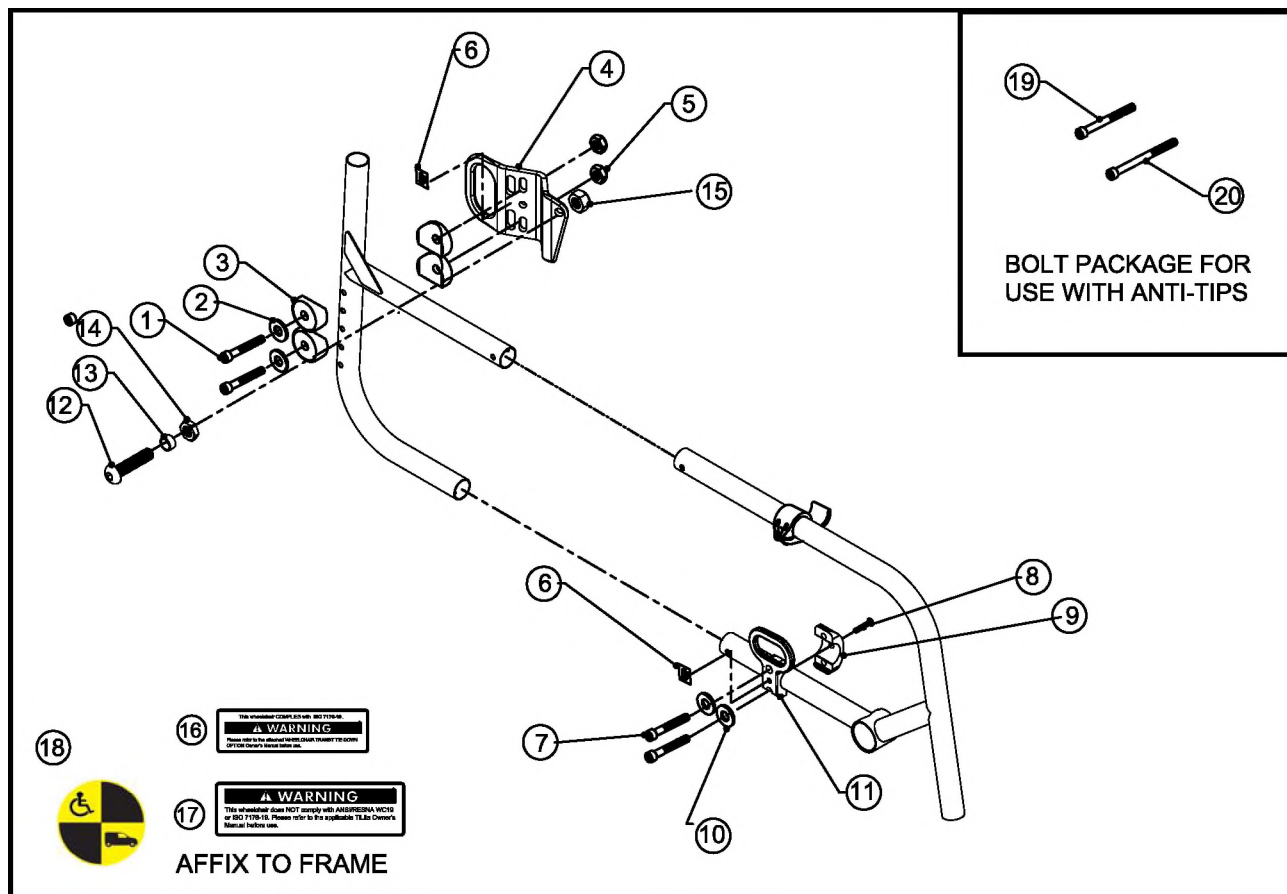

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	4	HD100563	1/4-28 X 1-3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
2	4	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
3	8	HD100014	COVERED WASHER, 1" OD X 1/4" ID X 1/8"
4A	1	CP107801	TIEDOWN ANCHOR MOUNT WITH BELT, LEFT
4B	1	CP107802	TIEDOWN ANCHOR MOUNT WITH BELT, RIGHT
5	4	HD100333	1/4-28 NYLOCK JAMNUT, BLACK ZINC
6	4	CP103109	DECAL, TRANSIT HOOK
7	4	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW
8	2	HD100123	1/4-20 X 1-1/2" BUTTON HEAD CAP SCREW
9	2	CP100519	C-CLAMP
10	4	HD100348	1/4" WASHER
11	2	CP107395	1" TIE DOWN BRACKET
12	2	HD100726	M8 X 35 MM BUTTON HEAD CAP SCREW
13	2	HD100717	M8 THIN NUT, BLACK ZINC
14	2	HD100719	NYLON LAP BELT SPACER
15	2	HD100718	M8 LOCK NUT, BLACK ZINC
16	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
17	1	RW100344	TRANSPORT BRACKET DECAL
18	2	RW100343	TRANSIT TIE-DOWN NOTIFICATION DECAL
19	4	HD100757	1/4-28 X 2" SOCKET HEAD CAP SCREW, ZINC PLATE
20	4	HD100758	1/4-28 X 2-3/4" SOCKET HEAD CAP SCREW, ZINC PLATE
1-16,18		AS118573	AERO X (SERIES 2) TRANSIT BRACKET PACKAGE
1-15,17		AS118574	2GX (SERIES 2) / AERO X (SERIES 2) TRANSPORT BRACKET PACKAGE
19-20		AS118581	AERO X (SERIES 2) TRANSIT BRACKET BOLT PACKAGE

TRANSIT AND TRANSPORT BRACKET OPTIONS


TiLite™ DEPTH ADJUSTABLE 2GX (SERIES 2), DEPTH ADJUSTABLE AERO X (SERIES 2)

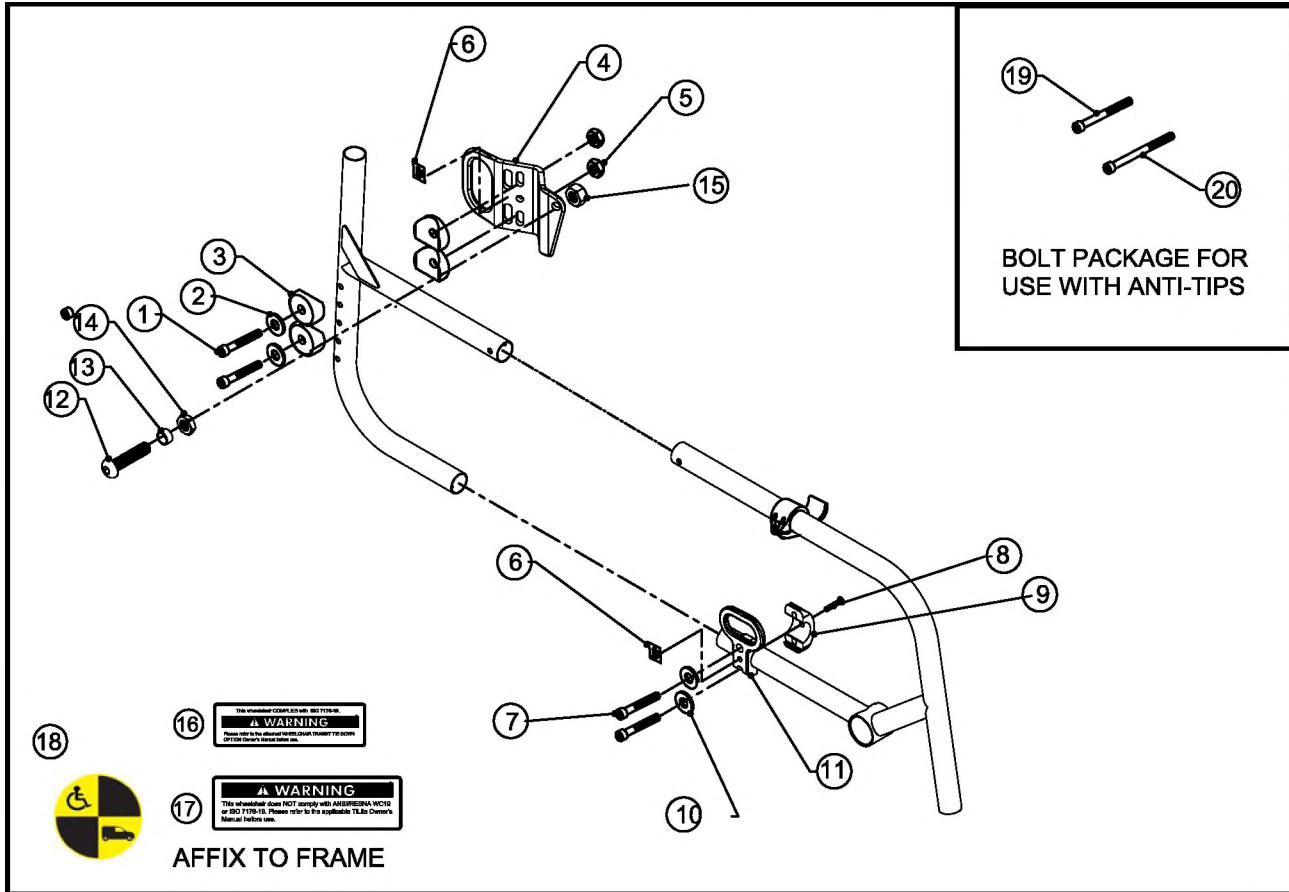

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	4	HD100563	1/4-28 X 1-3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
2	4	HD100454	1/4" SAE FLAT WASHER, BLACK ZINC
3	8	HD100014	COVERED WASHER, 1" OD X 1/4" ID X 1/8"
4A	1	CP107801	TIEDOWN ANCHOR MOUNT WITH BELT, LEFT
4B	1	CP107802	TIEDOWN ANCHOR MOUNT WITH BELT, RIGHT
5	4	HD100333	1/4-28 NYLOCK JAMNUT, BLACK ZINC
6	4	CP103109	DECAL, TRANSIT HOOK
7	4	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW
8	2	HD100361	1/4-20 X 1-1/8" BUTTON HEAD CAP SCREW
9	2	CP100519	C-CLAMP
10	4	HD100348	1/4" WASHER
11	2	CP107395	1" TIE DOWN BRACKET
12	2	HD100726	M8 X 35 MM BUTTON HEAD CAP SCREW
13	2	HD100717	M8 THIN NUT, BLACK ZINC
14	2	HD100719	NYLON LAP BELT SPACER
15	2	HD100718	M8 LOCK NUT, BLACK ZINC
16	1	RW100345	TRANSIT TIE-DOWN WARNING BRACKET DECAL
17	1	RW100344	TRANSPORT BRACKET DECAL
18	2	RW100343	TRANSIT TIE-DOWN NOTIFICATION DECAL
19	4	HD100757	1/4-28 X 2" SOCKET HEAD CAP SCREW, ZINC PLATE
20	4	HD100758	1/4-28 X 2-3/4" SOCKET HEAD CAP SCREW, ZINC PLATE
1-16,18		AS118595	AERO X (SERIES 2) TRANSIT BRACKET PACKAGE
1-15,17		AS118596	2GX (SERIES 2) / AERO X (SERIES 2) TRANSPORT BRACKET PACKAGE
19-20		AS118581	AERO X (SERIES 2) TRANSIT BRACKET BOLT PACKAGE

CATCH ALL


TiLite™ ZR, ZRA, TR(SERIES 2), YR(SERIES 2), AERO Z, Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2)

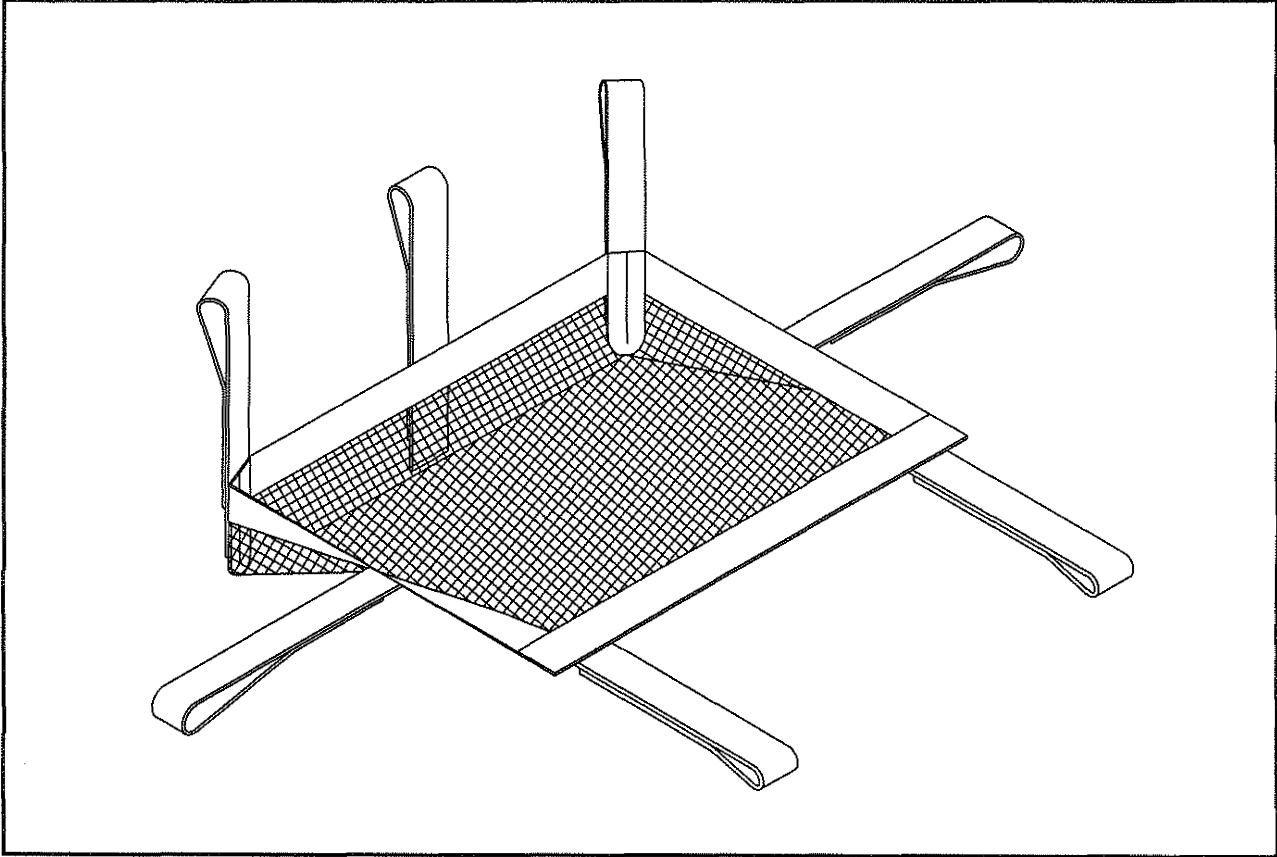

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103640	CATCH ALL, SMALL (14"-16" WIDTH)
1B	1	CP103641	CATCH ALL, MEDIUM (16"-18" WIDTH)
1C	1	CP103642	CATCH ALL, LARGE (18"-20" WIDTH)

CRUTCH HOLDER


TiLite™ ZR, ZRA, Aero Z, Aero R, TR (SERIES 2), YR (SERIES 2), TR, YR, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TWIST, TR (SERIES 3), AERO T, TRA (SERIES 2)

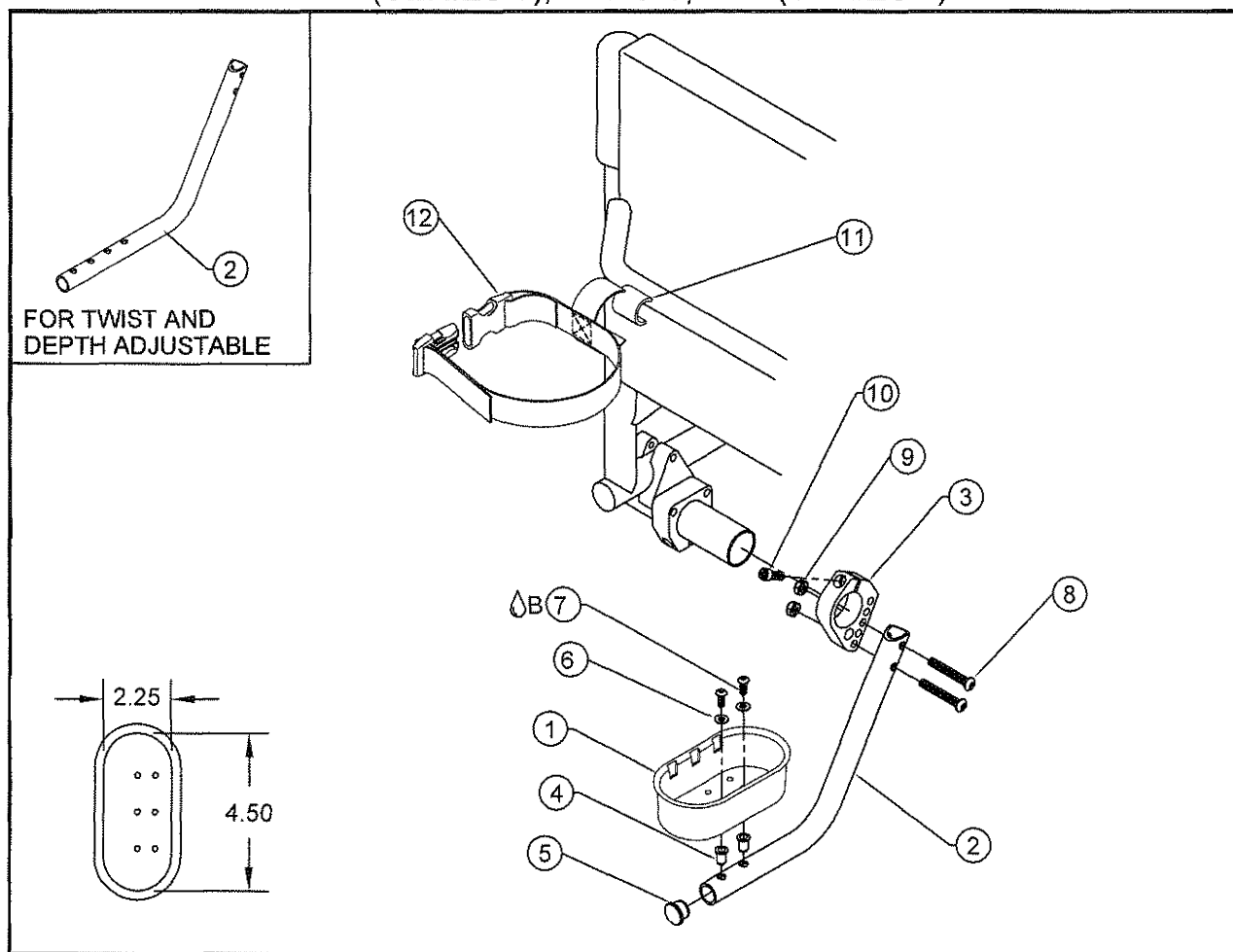

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103853	CRUTCH HOLDER CUP
2A	1	CP103770	CRUTCH HOLDER BENT TUBE
2B	1	CP105216	CRUTCH HOLDER BENT TUBE PED (FOR TWIST AND DEPTH ADJ)
3	1	CP102280	CAMBER CLAMP
4	2 OR 4	HD100154	RIVNUT, 10-32 X .130 - .225 GRIP RANGE
5	1	HD100127	END CAP 3/4" X .083 WALL TUBING
6	2	HD100289	#10 AN WASHER LIGHT SERIES
7 ^{ΔB}	2	HD100071	10-32 X 3/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
8	2	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
9	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
10	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
11	1"	RW100021	2" HOOK VELCRO ADHESIVE BACKED, BLACK
12	1	CP103771	CRUTCH HOLDER VELCRO STRAP FOR RIGID CHAIR
1,2A-12		AS104190	CRUTCH HOLDER FOR RIGID CHAIR
1,2B-12		AS111921	CRUTCH HOLDER FOR DEPTH ADJ CHAIR (FOR TWIST AND DEPTH ADJ)

CRUTCH HOLDER

TiLite™ X, SX, TX, 2GX, Aero X, YG, YGS, Aero X (SERIES 2), 2GX (SERIES 2)

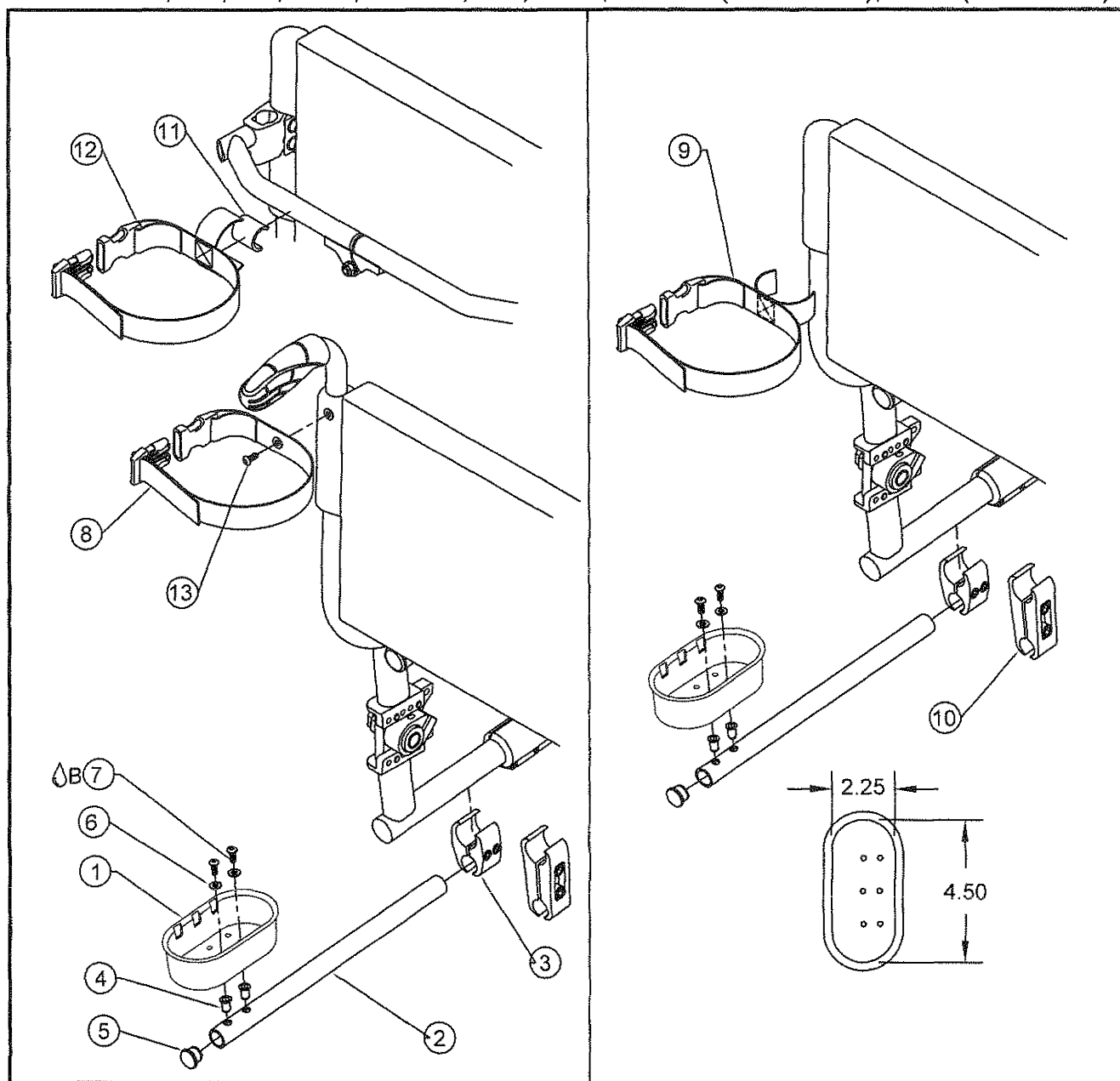


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103853	CRUTCH HOLDER CUP
2	1	CP103769	CRUTCH HOLDER STRAIGHT TUBE
3	1	CP100048	WHEEL LOCK CLAMP 1" TO 3/4"
4	2	HD100154	RIVNUT, 10-32 X .130 - .225 GRIP RANGE
5	1	HD100127	END CAP 3/4" X .083 WALL TUBING
6	2	HD100289	#10 AN WASHER LIGHT SERIES
7A	2	HD100071	10-32 X 3/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
8	1	CP103773	CRUTCH HOLDER VELCRO STRAP W/ GROMMET
9	1	CP103772	CRUTCH HOLDER VELCRO STRAP FOR FOLDING CHAIR
*10	1	AS103179	EXTENDED WHEELLOCK CLAMP FOR Ø1.00
11	1	RW100021	2" HOOK VELCRO ADHESIVE BACKED, BLACK
12	1	CP103771	CRUTCH HOLDER VELCRO STRAP FOR RIGID CHAIR
13A			BOLT THAT COMES WITH STANDARD NON-FOLDING BACKREST W/ PUSH HANDLES
13B	1	HD100736	REQUIRED BOLT USED WITH FOLD DOWN BACKREST W/ PUSH HANDLES
1-8		AS104191	CRUTCH HOLDER FOR FOLDING CHAIR W/ PUSH HANDLES
1-7,9		AS104192	CRUTCH HOLDER FOR FOLDING CHAIR W/O PUSH HANDLES
1-7,11,12		AS110366	CRUTCH HOLDER STRAP FOR FOLDING STABILIZER BAR

* ITEM 10 MAY BE USED TO PROVIDE CLEARANCE FOR ACCESSORIES INSTALLED ON CHAIR FRAME

UPPER EXTREMITY SUPPORT TRAY

ALL TiLite™ CHAIRS

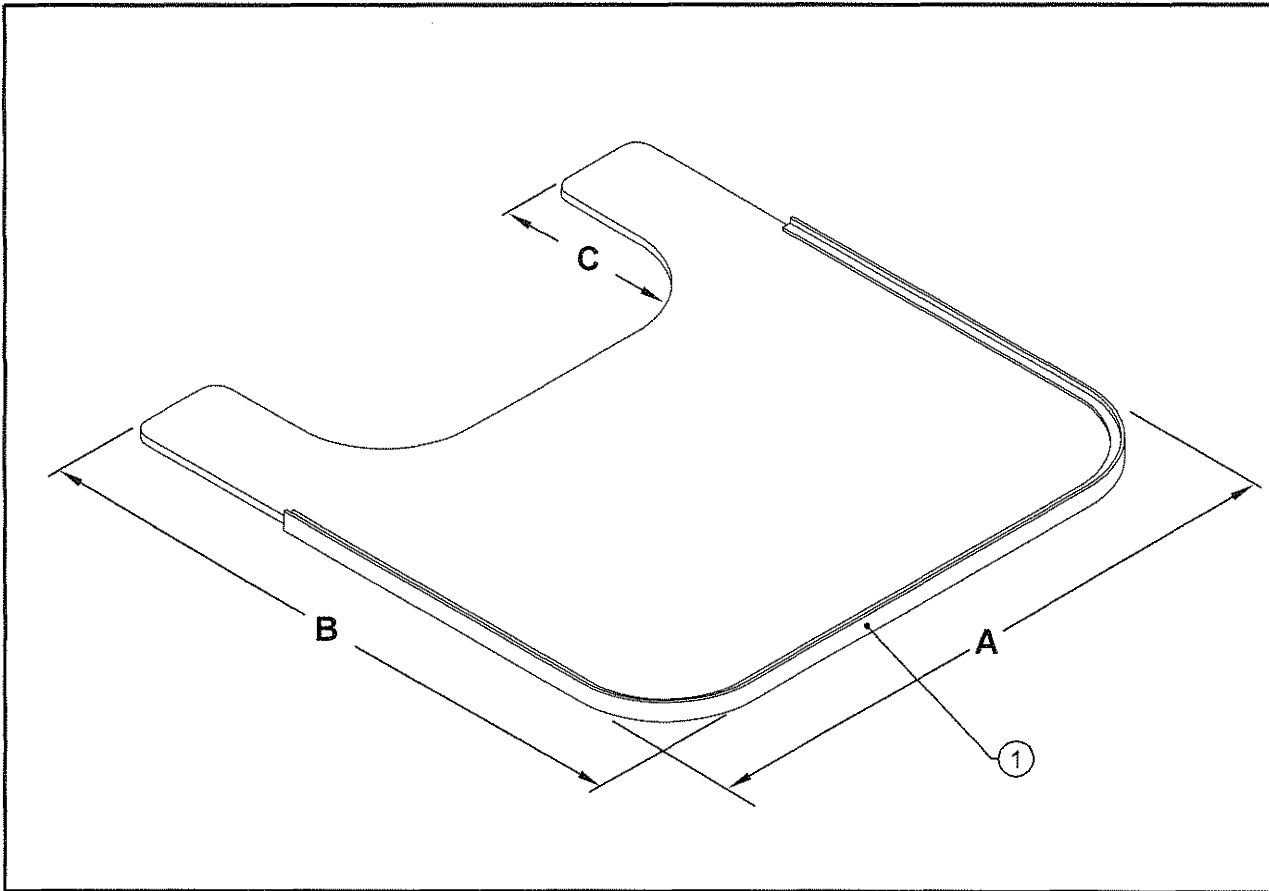


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP105262	UPPER EXTREMITY SUPPORT TRAY, 12"-14" SEAT WIDTHS
1B	1	CP105263	UPPER EXTREMITY SUPPORT TRAY, 13"-15" SEAT WIDTHS

TABLE ONE - TRAY DIMENSIONS

PART NUMBER	DIM "A"	DIM "B"	DIM "C"
CP105262	18.25"	19.25"	6"
CP105263	20.25"	20.25"	7"

HEX KEY SET

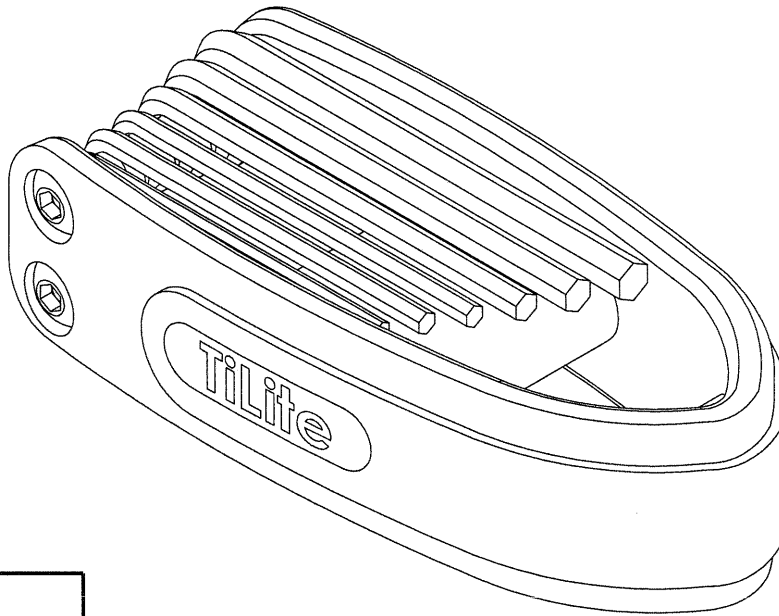

**9 PIECE SAE
HEX KEY SET**

5/64", 3/32", 7/64", 1/8"
9/64", 5/32", 3/16", 7/32", 1/4"

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP104895	SAE HEX KEY SET