

Case of Excellence: Nora
APPENDIX - Reference list

References

1. Arnold, A. J., Haworth, J. L., Moran, V. O., Abulhasan, A., Steinbuch, N., & Kokkoni, E. (2020). Exploring the Unmet Need for Technology to Promote Motor Ability in Children Younger Than 5 Years of Age: A Systematic Review. *Archives of Rehabilitation Research and Clinical Translation*, 2(2), 100051. <https://doi.org/10.1016/j.arrct.2020.100051>
- Chung, J., Evans, J., Lee, C., Lee, J., Rabbani, Y., Roxborough, L., & Harris, S. R. (2008). Effectiveness of adaptive seating on sitting posture and postural control in children with cerebral palsy. *Pediatric Physical Therapy*, 20(4), 303–317. <https://doi.org/10.1097/PEP.0b013e31818b7bdd>
- Huang, H. H., & Galloway, J. C. (2012). Modified ride-on toy cars for early power mobility: A technical report. *Pediatric Physical Therapy*, 24(2), 149–154. <https://doi.org/10.1097/PEP.0b013e31824d73f9>
- Livingstone, R. W., Bone, J., & Field, D. A. (2020). Beginning power mobility: An exploration of factors associated with child use of early power mobility devices and parent device preference. *Journal of rehabilitation and assistive technologies engineering*, 7, 2055668320926046.
- Logan, S. W., Catena, M. A., Sabet, A., Hospodar, C. M., Yohn, H., Govindan, A., & Galloway, J. C. (2019). Standing tall: Feasibility of a modified ride-on car that encourages standing. *Pediatric physical therapy*, 31(1), E6-E13.
- Stavness, C. (2006). The effect of positioning for children with cerebral palsy on upper-extremity function: a review of the evidence. *Physical & occupational therapy in pediatrics*, 26(3), 39-53.
2. Arps, K., Darr, N., & Katz, J. (2021). Effect of adapted motorized ride-on toy use on developmental skills, quality of life, and driving competency in nonambulatory children age 9–60 months. *Assistive technology*, 1-11.
- Hospodar, C. M., Sabet, A., Logan, S. W., Catena, M. A., & Galloway, J. C. (2021). Exploratory analysis of a developmentally progressive modified ride-on car intervention for young children with Down syndrome. *Disability and Rehabilitation: Assistive Technology*, 16(7), 749-757.
- Livingstone (A), R., & Field, D. (2014). Systematic review of power mobility outcomes for infants, children and adolescents with mobility limitations. *Clinical Rehabilitation*, 28(10), 954–964. <https://doi.org/10.1177/0269215514531262>
- Livingstone (B), R., & Field, D. (2015). The child and family experience of power mobility: A qualitative synthesis. *Developmental Medicine and Child Neurology*, 57(4), 317–327. <https://doi.org/10.1111/dmcn.12633>
3. Nilsson, L., Eklund, M., Nyberg, P., & Thulesius, H. (2011). Driving to learn in a powered wheelchair: the process of learning joystick use in people with profound cognitive disabilities. *American Journal of Occupational Therapy*, 65(6), 652-660.
- Stavness, C. (2006). The effect of positioning for children with cerebral palsy on upper-extremity function: a review of the evidence. *Physical & occupational therapy in pediatrics*, 26(3), 39-53.

References

4. Aceros, J., & Lundy, M. (2020). Enhanced Steering and Drive Adaptations of Modified Ride-On Toy Cars for Improved Directional Young Children With Severe Multiple Developmental Impairments. *Frontiers in Pediatrics*, 8(September).
- Fox, S., Guffey, H., O'Toole, L., Plummer, T. & Bernstein, J (2022 manuscript). Clinical Perspectives on the Explorer Mini.
- Logan, S. W., Hospodar, C. M., Feldner, H. A., Huang, H. H., & Galloway, J. C. (2018). Modified ride-on car use by young children with disabilities. *Pediatric physical therapy*, 30(1), 50-56.
- Rosen, L., Plummer, T., Sabet, A., Lange, M. L., & Livingstone, R. (2018). RESNA position on the application of power mobility devices for pediatric users. *Assistive Technology*.
5. Aceros, J., & Lundy, M. (2020). Enhanced steering and drive adaptations of modified ride-on toy cars for improved directional control in very young children with severe multiple developmental impairments. *Frontiers in pediatrics*, 8.
- Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini: Infants ' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17.
6. Mockler, S. R., McEwen, I. R., & Jones, M. A. (2017). Retrospective analysis of predictors of proficient power mobility in young children with severe motor impairments. *Archives of physical medicine and rehabilitation*, 98(10), 2034-2041.
- Nilsson, L., Eklund, M., Nyberg, P., & Thulesius, H. (2011). Driving to learn in a powered wheelchair: the process of learning joystick use in people with profound cognitive disabilities. *American Journal of Occupational Therapy*, 65(6), 652-660.
- Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini: Infants' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17.
- Pritchard-Wiart, L., Bragg, E., & Thompson-Hodgetts, S. (2019). The Young Movers Project: a case series describing modified toy car use as an early movement option for young children with mobility limitations. *Physical & occupational therapy in pediatrics*, 39(6), 598-613.

References

7. Arps, K., Darr, N., & Katz, J. (2021). Effect of adapted motorized ride-on toy use on developmental skills, quality of life, and driving competency in non-ambulatory children age 9–60 months. *Assistive technology*, 1-11.
- Bray, N., Kolehmainen, N., McAnuff, J., Tanner, L., Tuersley, L., Beyer, F., Grayston, A., Wilson, D., Edwards, R.T., Noyes, J., Craig, D. (2020). Powered mobility interventions for very young children with mobility limitations to aid participation and positive development: the EMPoWER evidence synthesis. 24(50).
- Huang, H. H., Chen, Y. M., Huang, H. W., Shih, M. K., Hsieh, Y. H., & Chen, C. L. (2018). Modified ride-on cars and young children with disabilities: effects of combining mobility and social training. *Frontiers in pediatrics*, 5, 299.
- Kenyon (D), L. K., Jones, M., Breaux, B., Tsotsoros, J., Gardner, T., & Livingstone, R. (2020). American and Canadian therapists' perspectives of age and cognitive skills for paediatric power mobility: a qualitative study. *Disability and Rehabilitation: Assistive Technology*, 15(6), 692–700. <https://doi.org/10.1080/17483107.2019.1606858>
- Livingstone (B), R., & Field, D. (2015). The child and family experience of power mobility: A qualitative synthesis. *Developmental Medicine and Child Neurology*, 57(4), 317–327.
- Livingstone (C), R., & Paleg, G. (2014). Practice considerations for the introduction and use of power mobility for children. *Developmental Medicine and Child Neurology*, 56(3), 210–221.
- Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini : Infants ' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17.
- Rosen, L., Plummer, T., Sabet, A., Lange, M. L., & Livingstone, R. (2018). RESNA position on the application of power mobility devices for pediatric users. *Assistive Technology*.
8. Johnson, E. W., & Yarnell, S. K. (1976). Hand dominance and scoliosis in Duchenne muscular dystrophy. *Archives of physical medicine and rehabilitation*, 57(10), 462-464.
- Nilsson, L., Eklund, M., Nyberg, P., & Thulesius, H. (2011). Driving to learn in a powered wheelchair: the process of learning joystick use in people with profound cognitive disabilities. *American Journal of Occupational Therapy*, 65(6), 652-660.

References

9. Craig, J., Hilderman, C., Wilson, G., & Misovic, R. (2016). Effectiveness of stretch interventions for children with neuromuscular disabilities: Evidence-based recommendations. *Pediatric Physical Therapy*, 28(3), 262–275. <https://doi.org/10.1097/PEP.0000000000000269>
- Glickman, L. B., Geigle, P. R., & Paleg, G. S. (2010). A systematic review of supported standing programs. *Journal of Pediatric Rehabilitation Medicine*, 3(3), 197–213. <https://doi.org/10.3233/PRM-2010-0129>
- Macias-Merlo, L., Bagur-Calafat, C., Girabent-Farrés, M., & A Stuberg, W. (2016). Effects of the standing program with hip abduction on hip acetabular development in children with spastic diplegia cerebral palsy. *Disability and Rehabilitation*, 38(11), 1075–108
- Macias-Merlo, L., Bagur-Calafat, C., Girabent-Farres, M., & Stuberg, W. A. (2015). Standing Programs to Promote Hip Flexibility in Children with Spastic Diplegic Cerebral Palsy. *Pediatric Physical Therapy*, 27(3), 243– 249
- Paleg, G. S., Smith, B. A., & Glickman, L. B. (2013). Systematic review and evidence-based clinical recommendations for dosing of pediatric supported standing programs. In *Pediatric Physical Therapy* (Vol. 25, Issue 3, pp. 232–247). <https://doi.org/10.1097/PEP.0b013e318299d5e7>
- Pountney, T. E., Mandy, A., Green, E., & Gard, P. R. (2009). Hip subluxation and dislocation in cerebral palsy—a prospective study on the effectiveness of postural management programmes. *Physiotherapy Research International*, 14(2), 116-127.

References

10. Craig, J., Hilderman, C., Wilson, G., & Misovic, R. (2016). Effectiveness of stretch interventions for children with neuromuscular disabilities: Evidence-based recommendations. *Pediatric Physical Therapy*, 28(3), 262–275. <https://doi.org/10.1097/PEP.0000000000000269>
- Glickman, L. B., Geigle, P. R., & Paleg, G. S. (2010). A systematic review of supported standing programs. *Journal of Pediatric Rehabilitation Medicine*, 3(3), 197–213. <https://doi.org/10.3233/PRM-2010-0129>
- Han, E. Y., Choi, J. H., Kim, S. H., & Im, S. H. (2017). The effect of weight bearing on bone mineral density and bone growth in children with cerebral palsy. *Medicine (United States)*, 96(10), 16–19.
- Paleg, G. S., Smith, B. A., & Glickman, L. B. (2013). Systematic review and evidence-based clinical recommendations for dosing of pediatric supported standing programs.
11. Hospodar, C. M., Sabet, A., Logan, S. W., Catena, M. A., & Galloway, J. C. (2021). Exploratory analysis of a developmentally progressive modified ride-on car intervention for young children with Down syndrome. *Disability and Rehabilitation: Assistive Technology*, 16(7), 749-757.
- Logan, S. W., Catena, M. A., Sabet, A., Hospodar, C. M., Yohn, H., Govindan, A., & Galloway, J. C. (2019). Standing tall: Feasibility of a modified ride-on car that encourages standing. *Pediatric physical therapy*, 31(1), E6-E13.
12. Arnold, A. J., Haworth, J. L., Moran, V. O., Abulhasan, A., Steinbuch, N., & Kokkoni, E. (2020). Exploring the Unmet Need for Technology to Promote Motor Ability in Children Younger Than 5 Years of Age: A Systematic Review. *Archives of Rehabilitation Research and Clinical Translation*, 2(2), 100051. <https://doi.org/10.1016/j.arrct.2020.100051>
- Bray, N., Kolehmainen, N., McAnuff, J., Tanner, L., Tuersley, L., Beyer, F., Grayston, A., Wilson, D., Edwards, R.T., Noyes, J., Craig, D. (2020). Powered mobility interventions for very young children with mobilitylimitations to aid participation and positive development: the EMPoWER evidence synthesis.24(50).
- Dunaway, S., Montes, J., O'Hagen, J., Sproule, D. M., Vivo, D. C. D., & Kaufmann, P. (2013). Independentmobility after early introduction of a power wheelchair in spinal muscular atrophy. *Journal of child neurology*,28(5), 576-582.
- Feldner, H. A., Logan, S. W., & Galloway, J. C. (2016). Why the time is right for a radical paradigm shift in early powered mobility: The role of powered mobility technology devices, policy and stakeholders. *Disability and Rehabilitation: Assistive Technology*, 11(2), 89–102. <https://doi.org/10.3109/17483107.2015.1079651>
- Fox, S., Guffey, H., O'Toole, L., Plummer, T. & Bernstein, J (2022 manuscript). Clinical Perspectives on the Explorer Mini.
- Hospodar, C. M., Feldner, H. A., & Logan, S. W. (2021). Active mobility, active participation: a systematic review of modified ride-on car use by children with disabilities. *Disability and Rehabilitation: Assistive Technology*, 0(0), 1–15

References by group

12. Continued

Kenyon (G), L. K., Mortenson, W. Ben, & Miller, W. C. (2018). 'Power in Mobility': parent and therapist perspectives of the experiences of children learning to use powered mobility. *Developmental Medicine and Child Neurology*, 60(10), 1012–1017. <https://doi.org/10.1111/dmcn.13906>

Livingstone (A), R., & Field, D. (2014). Systematic review of power mobility outcomes for infants, children and adolescents with mobility limitations. *Clinical Rehabilitation*, 28(10), 954–964. <https://doi.org/10.1177/0269215514531262>

Livingstone (B), R., & Field, D. (2015). The child and family experience of power mobility: A qualitative synthesis. *Developmental Medicine and Child Neurology*, 57(4), 317–327.

Livingstone (C), R., & Paleg, G. (2014). Practice considerations for the introduction and use of power mobility for children. *Developmental Medicine and Child Neurology*, 56(3), 210–221.

Logan (D), S. W., Hospodar, C. M., Feldner, H. A., Huang, H. H., & Galloway, J. C. (2018). Modified Ride-On Car Use by Young Children with Disabilities. *Pediatric Physical Therapy*, 30(1), 50–56.

Paguinto, S.G., Kasparian, N.A., Bray, P., Farrar, M. (2019). Parents' perceptions of power wheelchair prescription for children with a neuromuscular disorder: A scoping review. *Category*. 61(02), 1–26.

Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini: Infants' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17.

Rosen, L., Plummer, T., Sabet, A., Lange, M. L., & Livingstone, R. (2018). RESNA position on the application of power mobility devices for pediatric users. *Assistive Technology*.

13. Livingstone (A), R., & Field, D. (2014). Systematic review of power mobility outcomes for infants, children and adolescents with mobility limitations. *Clinical Rehabilitation*, 28(10), 954–964. <https://doi.org/10.1177/0269215514531262>

Kenyon, L. K., Massingill, B., & Farris, J. P. (2021). Using a child's power mobility learner group to tailor power mobility interventions: a case series. *Disability and Rehabilitation: Assistive Technology*, 1-7.

Nilsson, L., Eklund, M., Nyberg, P., & Thulesius, H. (2011). Driving to learn in a powered wheelchair: the process of learning joystick use in people with profound cognitive disabilities. *American Journal of Occupational Therapy*, 65(6), 652-660.

References

14. Aceros, J., & Lundy, M. (2020). Enhanced steering and drive adaptations of modified ride-on toy cars for improved directional control in very young children with severe multiple developmental impairments. *Frontiers in pediatrics*, 8.
- Bray, N., Kolehmainen, N., McAnuff, J., Tanner, L., Tuersley, L., Beyer, F., Grayston, A., Wilson, D., Edwards, R.T., Noyes, J., Craig, D. (2020). Powered mobility interventions for very young children with mobility limitations to aid participation and positive development : the EMPoWER evidence synthesis. 24(50).
- Bray, N., Kolehmainen, N., McAnuff, J., Tanner, L., Tuersley, L., Beyer, F., Grayston, A., Wilson, D., Edwards, R.T., Noyes, J., Craig, D. (2020). Powered mobility interventions for very young children with mobility limitations to aid participation and positive development : the EMPoWER evidence synthesis. 24(50).
- Fox, S., Guffey, H., O'Toole, L., Plummer, T. & Bernstein, J (2022 manuscript). Clinical Perspectives on the Explorer Mini.
- Guerette, P., Furumasu, J., & Tefft, D. (2013). The positive effects of early powered mobility on children's psychosocial and play skills. *Assistive Technology*, 25(1), 39–48.
- Hospodar, C. M., Feldner, H. A., & Logan, S. W. (2021). Active mobility, active participation: a systematic review of modified ride-on car use by children with disabilities. *Disability and Rehabilitation: Assistive Technology*
- Livingstone (C), R., & Paleg, G. (2014). Practice considerations for the introduction and use of power mobility for children. *Developmental Medicine and Child Neurology*, 56(3), 210–221.
- Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini: Infants ' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17
- Restrepo, P., Velásquez, J., Múnera, S., & Quintero Valencia, C. A. (2021). Adapting ride-on toy cars as a tool to promote leisure: A feasibility study in Colombia. *Assistive Technology*, 33(4), 217–222. <https://doi.org/10.1080/10400435.2019.1611675>
- Rosen, L., Plummer, T., Sabet, A., Lange, M. L., & Livingstone, R. (2018). RESNA position on the application of power mobility devices for pediatric users. *Assistive Technology*.
- Ross, S. M., Catena, M., Twardzik, E., Hospodar, C., Cook, E., Ayyagari, A., ... & Logan, S. W. (2018). Feasibility of a modified ride-on car intervention on play behaviors during an inclusive playgroup. *Physical & occupational therapy in pediatrics*, 38(5), 493-509.
15. Pritchard-Wiart, L., Bragg, E., & Thompson-Hodgetts, S. (2019). The Young Movers Project: a case series describing modified toy car use as an early movement option for young children with mobility limitations. *Physical & occupational therapy in pediatrics*, 39(6), 598-613.

References

16. Arps, K., Darr, N., & Katz, J. (2021). Effect of adapted motorized ride-on toy use on developmental skills, quality of life, and driving competency in non-ambulatory children age 9–60 months. *Assistive technology*, 1-11.
- Bray, N., Kolehmainen, N., McAnuff, J., Tanner, L., Tuersley, L., Beyer, F., Grayston, A., Wilson, D., Edwards, R.T., Noyes, J., Craig, D. (2020). Powered mobility interventions for very young children with mobility limitations to aid participation and positive development: the EMPoWER evidence synthesis. 24(50).
- Fox, S., Guffey, H., O'Toole, L., Plummer, T. & Bernstein, J (2022 manuscript). *Clinical Perspectives on the Explorer Mini*.
- Guerette, P., Furumasu, J., & Tefft, D. (2013). The positive effects of early powered mobility on children's psychosocial and play skills. *Assistive Technology*, 25(1), 39–48.
- Huang, H. H. (2018). Perspectives on early power mobility training, motivation, and social participation in young children with motor disabilities. *Frontiers in Psychology*, 8(JAN), 1–8. <https://doi.org/10.3389/fpsyg.2017.02330>
- Huang, H. H., Chen, Y. M., Huang, H. W., Shih, M. K., Hsieh, Y. H., & Chen, C. L. (2018). Modified ride-on cars and young children with disabilities: effects of combining mobility and social training. *Frontiers in pediatrics*, 5, 299.
- Kenyon (D), L. K., Jones, M., Breaux, B., Tsotsoros, J., Gardner, T., & Livingstone, R. (2020). American and Canadian therapists' perspectives of age and cognitive skills for paediatric power mobility: a qualitative study. *Disability and Rehabilitation: Assistive Technology*, 15(6), 692–700. <https://doi.org/10.1080/17483107.2019.1606858>
- Livingstone (A), R., & Field, D. (2014). Systematic review of power mobility outcomes for infants, children and adolescents with mobility limitations. *Clinical Rehabilitation*, 28(10), 954–964. <https://doi.org/10.1177/0269215514531262>
- Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini: Infants' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17.
- Pritchard-Wiart, L., Bragg, E., & Thompson-Hodgetts, S. (2019). The Young Movers Project: a case series describing modified toy car use as an early movement option for young children with mobility limitations. *Physical & occupational therapy in pediatrics*, 39(6), 598-613.
- Rosen, L., Plummer, T., Sabet, A., Lange, M. L., & Livingstone, R. (2018). RESNA position on the application of power mobility devices for pediatric users. *Assistive Technology*.
- Ross, S. M., Catena, M., Twardzik, E., Hospodar, C., Cook, E., Ayyagari, A., ... & Logan, S. W. (2018). Feasibility of a modified ride-on car intervention on play behaviors during an inclusive playgroup. *Physical & occupational therapy in pediatrics*, 38(5), 493-509.

References

17. Arps, K., Darr, N., & Katz, J. (2021). Effect of adapted motorized ride-on toy use on developmental skills, quality of life, and driving competency in nonambulatory children aged 9–60 months. *Assistive technology*, 1-11.
- Kenyon (G), L. K., Mortenson, W. Ben, & Miller, W. C. (2018). 'Power in Mobility': parent and therapist perspectives of the experiences of children learning to use powered mobility. *Developmental Medicine and Child Neurology*, 60(10), 1012–1017
- Livingstone (A), R., & Field, D. (2014). Systematic review of power mobility outcomes for infants, children and adolescents with mobility limitations. *Clinical Rehabilitation*, 28(10), 954–964. <https://doi.org/10.1177/0269215514531262>
- Livingstone (C), R., & Paleg, G. (2014). Practice considerations for the introduction and use of power mobility for children. *Developmental Medicine and Child Neurology*, 56(3), 210–221.
- Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini : Infants ' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17.
- Pritchard-Wiart, L., Bragg, E., & Thompson-Hodgetts, S. (2019). The Young Movers Project: a case series describing modified toy car use as an early movement option for young children with mobility limitations. *Physical & occupational therapy in pediatrics*, 39(6), 598-613.

References

18. Aceros, J., & Lundy, M. (2020). Enhanced steering and drive adaptations of modified ride-on toy cars for improved directional control in very young children with severe multiple developmental impairments. *Frontiers in pediatrics*, 8.
- Bray, N., Kolehmainen, N., McAnuff, J., Tanner, L., Tuersley, L., Beyer, F., Grayston, A., Wilson, D., Edwards, R.T., Noyes, J., Craig, D. (2020). Powered mobility interventions for very young children with mobility limitations to aid participation and positive development : the EMPoWER evidence synthesis. 24(50).
- Fox, S., Guffey, H., O'Toole, L., Plummer, T. & Bernstein, J (2022 manuscript). *Clinical Perspectives on the Explorer Mini*.
- Hospodar, C. M., Feldner, H. A., & Logan, S. W. (2021). Active mobility, active participation: a systematic review of modified ride-on car use by children with disabilities. *Disability and Rehabilitation: Assistive Technology*
- Huang, H. H. (2018). Perspectives on early power mobility training, motivation, and social participation in young children with motor disabilities. *Frontiers in Psychology*, 8(JAN), 1–8. <https://doi.org/10.3389/fpsyg.2017.02330>
- Livingstone (A), R., & Field, D. (2014). Systematic review of power mobility outcomes for infants, children and adolescents with mobility limitations. *Clinical Rehabilitation*, 28(10), 954–964.
- Livingstone (B), R., & Field, D. (2015). The child and family experience of power mobility: A qualitative synthesis. *Developmental Medicine and Child Neurology*, 57(4), 317–327. <https://doi.org/10.1111/dmcn.12633>
- Logan (C), S.W., Hospodar, C.M., Bogart, K.R., Catena, M.A., Feldner, H.A., Fitzgerald, J., Schaffer, S., Sloane, B., Phelps, B., Phelps, J., Smart, W.D. (2019). Real World Tracking of Modified Ride-On Car Usage in Young Children With Disabilities. *Physiology & Behavior*, 176(1), 100–106.
- Paguinto, S.G., Kasparian, N.A., Bray, P., Farrar, M. (2019). Parents' perceptions of power wheelchair prescription for children with a neuromuscular disorder: A scoping review. *Category*. 61(02), 1–26.
- Paguinto, S.G., Kasparian, N.A., Bray, P., Farrar, M. (2019). Parents' perceptions of power wheelchair prescription for children with a neuromuscular disorder: A scoping review. *Category*. 61(02), 1–26.
- Rosen, L., Plummer, T., Sabet, A., Lange, M. L., & Livingstone, R. (2018). RESNA position on the application of power mobility devices for pediatric users. *Assistive Technology*.
- Ross, S. M., Catena, M., Twardzik, E., Hospodar, C., Cook, E., Ayyagari, A., ... & Logan, S. W. (2018). Feasibility of a modified ride-on car intervention on play behaviors during an inclusive playgroup. *Physical & occupational therapy in pediatrics*, 38(5), 493-509.

References

19. Hospodar, C. M., Feldner, H. A., & Logan, S. W. (2021). Active mobility, active participation: a systematic review of modified ride-on car use by children with disabilities. *Disability and Rehabilitation: Assistive Technology*
- Kenyon, L. K., Farris, J. P., Gallagher, C., Hammond, L., Webster, L. M., & Aldrich, N. J. (2017). Power mobility training for young children with multiple, severe impairments: a case series. *Physical & occupational therapy in pediatrics*, 37(1), 19-34.
- Livingstone (F), R., Field, D., Sanderson, C., Pineau, N., & Zwicker, J. G. (2020). Beginning power mobility: parent and therapist perspectives. *Disability and Rehabilitation*, 0(0), 1–10. <https://doi.org/10.1080/09638288.2020.1842916>
- Logan, S. W., Catena, M. A., Sabet, A., Hospodar, C. M., Yohn, H., Govindan, A., & Galloway, J. C. (2019). Standing tall: Feasibility of a modified ride-on car that encourages standing. *Pediatric physical therapy*, 31(1), E6-E13.
- Logan, S. W., Feldner, H. A., Galloway, J. C., & Huang, H. H. (2016). Modified ride-on car use by children with complex medical needs. *Pediatric Physical Therapy*, 28(1), 100-107.
- Plummer, T., Logan, S. W., Morress, C., (2020). Explorer Mini : Infants ' Initial Experience with a Novel Pediatric Powered Mobility Device. *Physical & Occupational Therapy In Pediatrics*, 0(0), 1–17.
- Pritchard-Wiart, L., Bragg, E., & Thompson-Hodgetts, S. (2019). The Young Movers Project: a case series describing modified toy car use as an early movement option for young children with mobility limitations. *Physical & occupational therapy in pediatrics*, 39(6), 598-613.
- Restrepo, P., Velásquez, J., Múnera, S., & Quintero Valencia, C. A. (2021). Adapting ride-on toy cars as a tool to promote leisure: A feasibility study in Colombia. *Assistive Technology*, 33(4), 217–222.
20. Arnold, A. J., Haworth, J. L., Moran, V. O., Abulhasan, A., Steinbuch, N., & Kokkoni, E. (2020). Exploring the Unmet Need for Technology to Promote Motor Ability in Children Younger Than 5 Years of Age: A Systematic Review. *Archives of Rehabilitation Research and Clinical Translation*, 2(2), 100051. <https://doi.org/10.1016/j.arrct.2020.100051>
- Fox, S., Guffey, H., O'Toole, L., Plummer, T. & Bernstein, J (2022 manuscript). *Clinical Perspectives on the Explorer Mini*.
- Livingstone (A), R., & Field, D. (2014). Systematic review of power mobility outcomes for infants, children and adolescents with mobility limitations. *Clinical Rehabilitation*, 28(10), 954–964.
- Livingstone, R. W., Bone, J., & Field, D. A. (2020). Beginning power mobility: An exploration of factors associated with child use of early power mobility devices and parent device preference. *Journal of rehabilitation and assistive technologies engineering*, 7, 2055668320926046.
- Paguinto, S.G., Kasparian, N.A., Bray, P., Farrar, M. (2019). Parents' perceptions of power wheelchair prescription for children with a neuromuscular disorder: A scoping review. *Category*. 61(02), 1–26.
- Pritchard-Wiart, L., Bragg, E., & Thompson-Hodgetts, S. (2019). The Young Movers Project: a case series describing modified toy car use as an early movement option for young children with mobility limitations. *Physical & occupational therapy in pediatrics*, 39(6), 598-613.