

ROHO®

Hybrid Select Cushion

Exceptional positioning with
proven skin protection.



permobil

Advanced hybrid seating with exceptional positioning

The Hybrid Select pelvic loading area adjusts to your unique shape, weight, and positioning changes. The ROHO overlay and IT insert utilizes the ISOFLO® Memory Control that allows you to easily find and set the right position each and every time.



3-Zone ISOFLO Memory Control for adjustable positioning

Attributes of great seating

Mechanical testing



Skin Protection

Immersion test (ISO 16840-2) reveals how deeply the body sinks into the cushion.

Envelopment test (ISO 16840-12) reveals how the body weight is cradled by the cushion, using envelopment or offloading.

Shear test (Shear force sensor) reveals the potential of the cushion interaction with the body to distort/deform the tissue.

Contact Area (Pressure mapping) reveals how much contact is being made between the seat support and the individual.



Positioning

Slide resistant test (ISO 16840-2) reveals how much force is required to slide forward on the cushion when seated.



Stability


Lateral tilt test (ISO 16840-13) reveals the stability the cushion may provide during side-to-side leaning.



Unique ROHO cell design for skin protection and immersion




Pelvic well using patented ROHO DRY FLOATATION® technology for load distribution and 3.2 cm immersion.

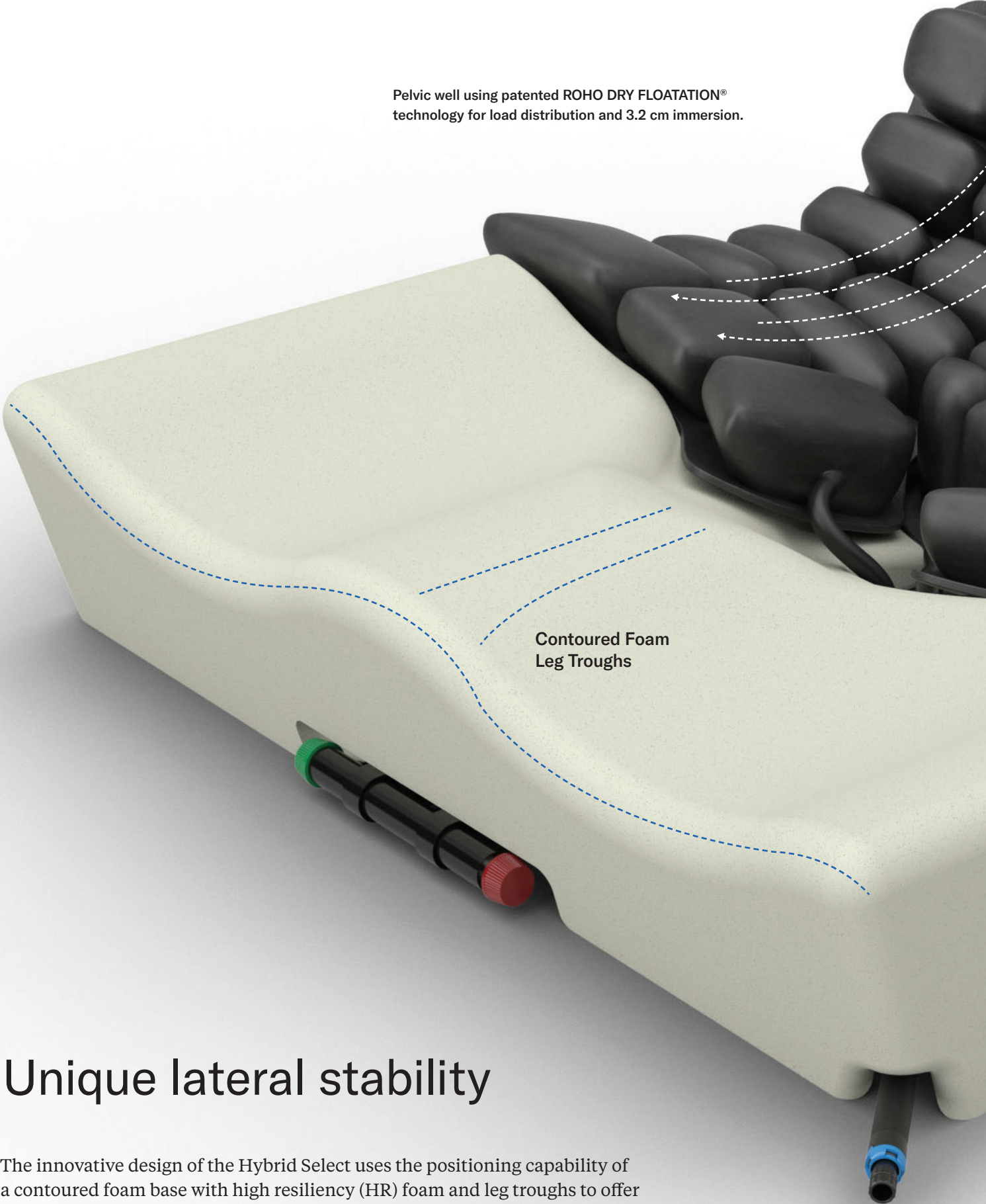

Contoured Foam
Leg Troughs

Unique lateral stability

The innovative design of the Hybrid Select uses the positioning capability of a contoured foam base with high resiliency (HR) foam and leg troughs to offer you support, lateral stability, and provide relief to the coccyx/tailbone.

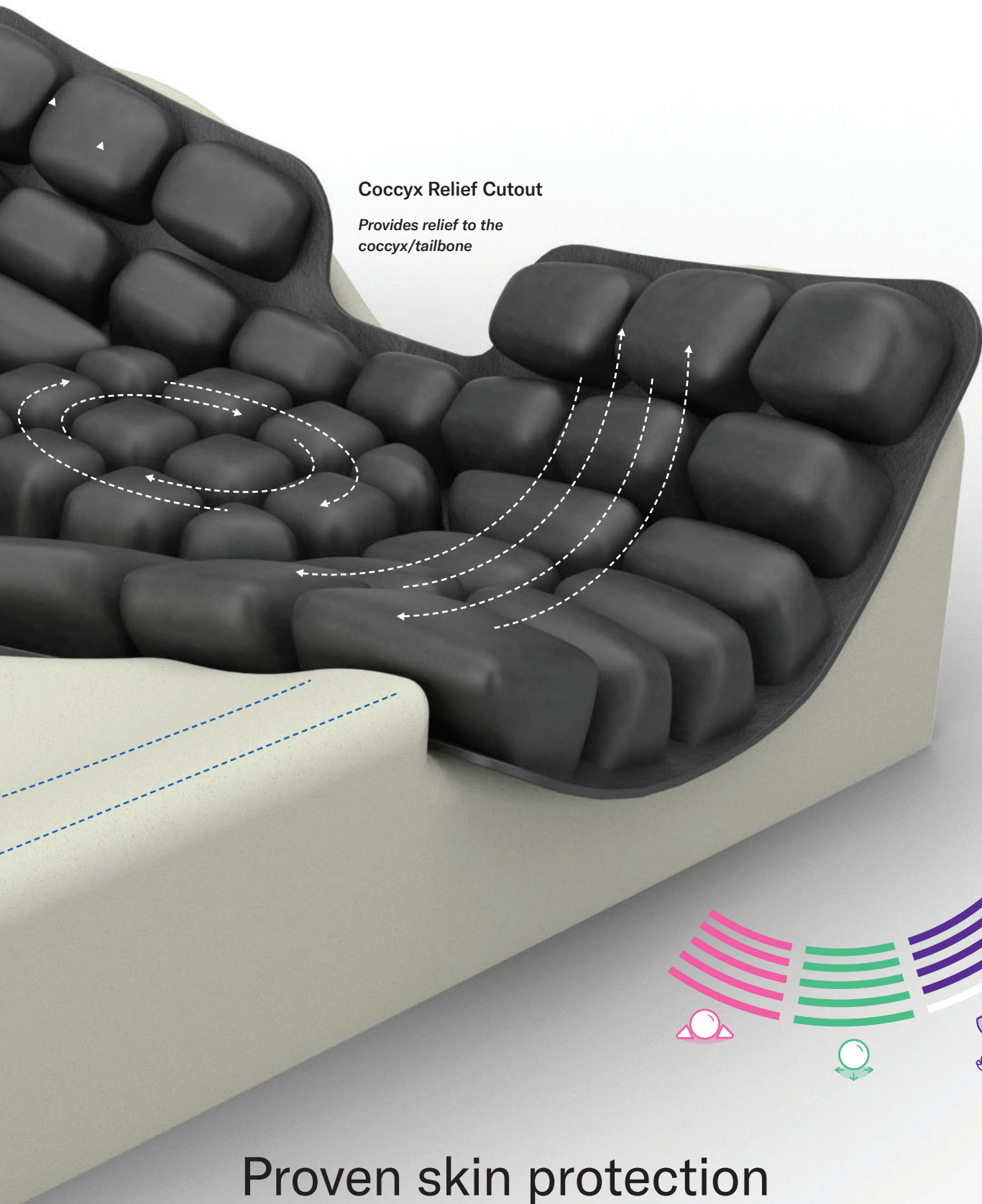

Coccyx Relief Cutout

Provides relief to the coccyx/tailbone

Proven skin protection

With Hybrid Select, experience truly functional stability combined with great skin and soft tissue protection. The contoured design of the Hybrid Select allows your pelvis to be immersed in air which shapes to you, moves with you, and protects your skin and soft tissue.

- Immersion
- Envelopment
- Offloading
- Positioning
- Stability
- Skin Protection

Optional offloading comes standard

The Hybrid Select provides flexibility and forgiveness that other offloading cushions do not, making it easier to be in the right position each and every time. The Hybrid Select offers an exclusive optional offloading technique by removing the ischial tuberosities (IT) air insert.



Offloading

*Removable IT air insert
for optional offloading*



The Hybrid Select gives me the positioning I need and makes it easier for me to transfer.

— Eric
Permobil User



Specifications

| | |
|------------------|------------------|
| Cover type | Stretch-Air® |
| Product weight | 1.45 kg* |
| Weight limit | 227 kg |
| Cushion warranty | 36 month limited |
| Cover warranty | 6 month limited |

*Product weight is based on a size 40.5 x 40.5 cushion. Product weight varies depending on cushion size.

Size options

| | | | |
|-------------------|-------------------|-------------------|-------------------|
| 35.5 cm x 37.5 cm | 40.5 cm x 42.5 cm | 43.0 cm x 47.5 cm | 51.0 cm x 42.5 cm |
| 35.5 cm x 42.5 cm | 40.5 cm x 45.0 cm | 43.0 cm x 52.5 cm | 51.0 cm x 45.0 cm |
| 38.0 cm x 40.0 cm | 40.5 cm x 47.5 cm | 45.5 cm x 42.5 cm | 51.0 cm x 47.5 cm |
| 38.0 cm x 42.5 cm | 40.5 cm x 52.5 cm | 45.5 cm x 45.0 cm | 51.0 cm x 52.5 cm |
| 38.0 cm x 45.0 cm | 43.0 cm x 42.5 cm | 45.5 cm x 47.5 cm | |
| 40.5 cm x 40.0 cm | 43.0 cm x 45.0 cm | 45.5 cm x 52.5 cm | |

The following are trademarks and registered trademarks of Permobil: ROHO®, ISOFLO®, Stretch-Air® and DRY FLOATATION®.