ROHO® Reusable Overlay Cover Instructions for Use

 For additional information about using this cover, refer to the instructions provided with a compatible ROHO Mattress Overlay.

Supplier: This manual must be given to the user of this product. Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.	Incident Reporting: In the event of a serious incident or injury, contact Customer Support. * ⚠ Warning Indicates that not following the specified instructions may lead to hazardous conditions resulting in serious injury.
---	---

Intended Use: The ROHO Reusable Overlay Cover is a one-piece fluid-resistant textile intended to protect a ROHO Mattress Overlay from solid debris or liquids.

Compatibility: Compatible mattress overlays are shown on the cover care label.

In addition to the intended use statement, a clinician should be consulted regarding use of this cover within the individual's overall care regimen and including any need for large-print instructions or other assistive technology.

Cover Placement

⚠ Warnings: Periodically, inspect the cover for damage and replace if necessary. • If you are unable to perform any steps, seek assistance. • The mattress overlay surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the overlay will contact unprotected skin. • DO NOT use the product as a transfer surface.

- Before the individual transfers to the bed, place the cover around the overlay and the bed mattress, referring to the directional labels to orient the cover.
- If needed, the following items may be placed on top of the Mattress Overlay and Reusable Overlay Cover: One flat or contoured sheet, arranged loosely • One incontinent pad • One draw sheet, for moving and transferring.

⚠ Warning: Placing too many liners between the individual and the product will reduce its effectiveness.

Cover Removal: The individual should transfer to a different surface. Pull each corner away from the bed mattress and overlay.

Cover Care

⚠ Warnings: Clean the product regularly and when it becomes soiled. If the product becomes soiled, or between use by different individuals: clean, disinfect, and check for proper functioning. • Follow all manufacturer bleach container and germicide/disinfectant safety guidelines. • This product does not require sterilization; DO NOT autoclave. • DO NOT dry clean. **✖**

Notes: Respeated washing at high temperatures can accelerate deterioration of the cover and is not recommended. • Quaternary-type disinfectants may also be used in concentrations recommended by the manufacturer.

Symbols Glossary									
REF	Model number	FITS	Fits		Date of manufacture		Manufacturer		Fall risk
MD	Medical Device	WDH	Size		Meets flammability		Machine wash hot, normal, at the temperature shown.		Hand wash.
LOT	Batch code		Consult instructions for use.		Authorized representative in the European Community		Machine wash cold, gentle.		Tumble dry low.
			Suffocation risk		Bleach (1 part bleach: 9 parts water).		Do not dry clean.		Cleaning instructions
			Disinfecting instructions						

Limited Warranty Term: 6 months; see also the Limited Warranty Supplement provided with the product, or contact Customer Support.

Product information is changed as needed; current product information is available at permobilus.com. • ROHO® is a registered trademark of Permobil.

• ROHO, Inc. is a part of Permobil Seating and Positioning.

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