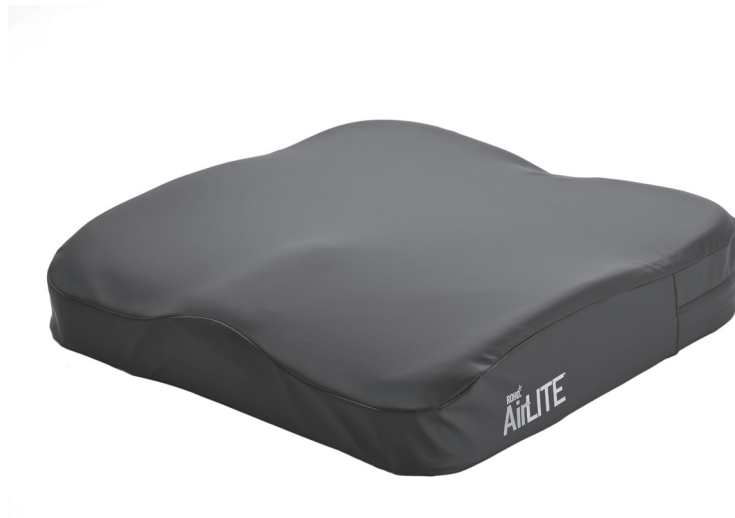
ROHO®

# AirLITE® Cushion

Operation Manual

**permobil**

s h a p e f i t t i n g t e c h n o l o g y®

**Contact Information**

 ROHO, Inc.  
 1501 South 74th Street  
 Belleville, IL 62223 U.S.A.  
 U.S.A.:  
 800-736-0925  
 Fax 888-551-3449  
 Customer Support:  
 orders.roho@permobil.com  
 Outside the U.S.A.:  
 618-277-9150  
 Fax 618-277-6518  
 intlorders.roho@permobil.com  
 permobil.com

 **Warning**

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

 **Caution**

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

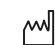
**Incident Reporting**

In the event of a serious incident or injury, contact Customer Support.

**Contents**

Intended Use			2
Important Safety Information			
Parts Detail and Product Specifications			3
Cushion Placement			
Cover Removal and Placement			
Troubleshooting			4
Storage, Transport, Disposal and Recycling			
Cleaning and Disinfecting			5
Limited Warranty			
Español	6	Dansk	31
Français	11	Svenska	36
Deutsch	16	Norsk	41
Italiano	21	Português	46
Nederlands	26	Polski	51

**Symbols Glossary**

 Manufacturer	 Fits	 Size (width x depth x height)	 Front of product
 Date of manufacture	 Model number	 Weight limit	 Back of product
 Consult instructions for use	 Serial number	 Medical Device	 Bottom of product
 Quantity	 Meets flammability	 Cleaning instructions	 Disinfecting instructions
 Batch code	 Authorized representative in the European Community	 Authorized representative in Switzerland	

**Please Note**

Supplier: This manual must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Products may be covered by one or more U.S. and foreign patents and trademarks, including AIR FLOATATION® Technology, AirLITE®, ROHO®, and shape fitting technology®.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

ROHO, Inc. is part of Permobil.

© 2014, 2022 Permobil

Rev. 2022-12-08

## Intended Use

The ROHO AirLITE Cushion (AirLITE) is a wheelchair support surface with a non-adjustable, air-filled, segmented-design, AIR FLOATION Technology insert, encased in contoured foam. The AirLITE is intended to conform to an individual's seated shape to provide positioning. There is a weight limit of 300 lbs. (136 kg), and the cushion must be properly sized to the individual. The AirLITE must be used with the supplied cover.

*A clinician who is experienced in seating and positioning should determine whether the product is appropriate for the individual's particular seating needs. The product's intended use statement alone should not be relied upon to make that determination.*

*The medical products featured in this document are intended to be only one part of an overall care regimen that includes all seating and mobility equipment and therapeutic interventions. A clinician who is experienced in seating and positioning should identify this care regimen after evaluating the individual's physical needs and overall medical condition.*

*A clinician should also evaluate for visual, reading, and cognitive impairment to determine the need for caregiver assistance or the need for other assistive technology, such as large-print instructions, to ensure proper product use.*

## Important Safety Information

### Warnings:

- Skin/soft tissue breakdown can occur due to a number of factors, which vary by individual. Check skin frequently, at least once a day. Redness, bruising, or darker areas (when compared to normal skin) may indicate superficial or deep tissue injury and should be addressed. If there is any discoloration to skin/soft tissue, STOP USE immediately. If the discoloration does not disappear within 30 minutes after disuse, immediately consult a healthcare professional.
- DO NOT use the cushion on top of, or in conjunction with, any other products or materials, except as indicated in this manual. Using noncompatible products or materials, whether inside or outside the cover: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.
- DO NOT place obstructions between the individual and the cushion. The cushion and the cover MUST be compatible sizes and MUST be used as directed in this manual. Except for the AirLITE Cover, placement of any items between the individual and the cushion: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.
- The AirLITE must be used with the provided cover.
- DO NOT expose the product to high heat, open flames or hot ashes. Testing or certification claims, including for flammability, may no longer apply to this device when it is combined with other products or materials. Check testing and certification claims for all products used in combination with this device.
- The cushion surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the cushion will contact unprotected skin.
- If you are unable to perform any tasks described in this manual, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.
- DO NOT use the cushion as a water flotation device (e.g., a Life Preserver). It will NOT support you in water.

### Cautions:

- DO NOT modify the cushion or any components. Doing so may result in damage to the product and will void the warranty.
- Periodically, check the cushion and components for damage and replace if necessary. See "Troubleshooting."

### Warnings - Motor Vehicle Transportation:

- Failure to pay attention to these warnings could result in severe injury to the individual in the wheelchair or to others.
- Whenever possible, transfer out of the wheelchair and into a manufacturer-installed vehicle seat without the cushion, and use the vehicle's crash-tested occupant restraint system.
- The cushion must be correctly and securely installed, used, and maintained according to the instructions and warnings provided by the cushion manufacturer.
- When the wheelchair is used as the vehicle seat, the cushion should only be used with wheelchair frames that are equipped with securement points specified in ANSI/RESNA WC-4:2017, section 19 (WC19) or ISO 7176-19, and that have been successfully crash tested on a wheelchair in the forward-facing position.
- The wheelchair with the cushion must be used with an effective wheelchair securement system and a properly positioned, crash-tested pelvic and shoulder-belt restraint, or Wheelchair Tiedown and Occupant Restraint System (WTORS), following the manufacturer's instructions.
- If the cushion has been involved in an accident during transport, check the cushion for damage and replace if necessary. See the "Troubleshooting" section in this manual.
- For more information about safely transporting wheelchair users, refer to: <http://wctransportation-safety.umtri.umich.edu/ridesafe-brochure>.
- A ROHO QUADTRO SELECT Cushion was included in a wheelchair seating system that was dynamically tested by a third-party testing facility for use in a motor vehicle. The system tested meets all applicable criteria for wheelchair seating systems proposed in Section 5.1 of ANSI/RESNA WC-4:2017 Section 20 "Wheelchair Seating Systems for Use in Motor Vehicles."
- Test results apply to seating systems intended for use by adults and children with a total body mass greater than 23 kg (51 lb, or approximately 6 years of age) who are transported facing forward in their wheelchairs in all types of motor vehicles.
- To determine whether motor vehicle third-party testing has been performed with a specific seating system combination, refer to all provided equipment operation manuals or contact the equipment manufacturer.

## Parts Detail and Product Specifications

**Package contents:** cushion, cover, operation manual, limited warranty supplement, product registration card.

To order replacement parts, contact the equipment provider or Customer Support. The serial number label is located on the bottom of the cushion.

**Materials:** Cushion: non-adjustable, segmented, polyurethane air insert encased in polyurethane and polyethylene foam; Cover: fluid-resistant, two-way stretch top, two-way stretch sides, and non-skid bottom; refer to cover care label for material content and for compliance to flammability requirements.

**Sizing** (All measurements are approximate.)

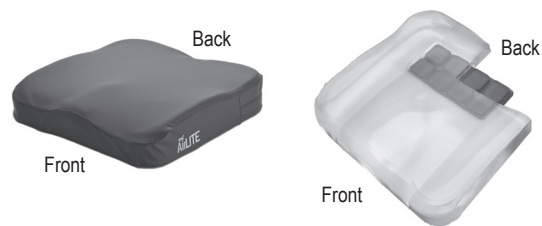
Smallest Width: 13 in. (33cm); Largest Width: 20 in. (51cm)

Smallest Depth: 13 in. (33cm); Largest Depth: 20 in. (51cm)

Average weight: 2 lb. (0.9kg)

Height ranges from 2 in. (5cm) to 3¼ in. (9.5cm)

1 in. = 2.54 cm = 25.4 mm

### Accessories Compatible with AirLITE:

Not available in all markets.

- ROHO AirLITE Convex Insert
- ROHO Planar Solid Seat Insert

**Expected lifetime:** cushion: 5 years; cover: 1 year

## Cushion Placement

**Warning:** When possible, use hook and loop fasteners to further secure the cushion to the seating surface. Hook fasteners are available upon request.

- 1 Make sure the cover is correctly installed on the cushion. See the instructions for cover placement.

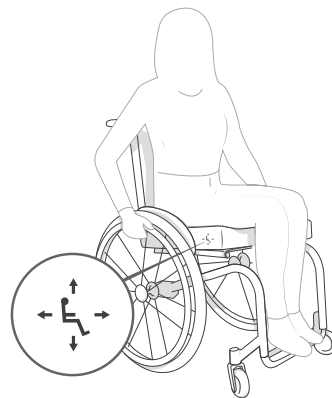
*Note: If an insert is used, place it inside the cover, beneath the foam base. Refer to the instructions provided with the insert. See "Product Specifications" for a list of compatible inserts.*

- 2 Place the cushion on the wheelchair seat.

*Note: The symbol on the cover showing the seated figure may be used to help orient cushion.*

- 3 The individual should transfer to the cushion and sit in their usual sitting position.

- Confirm that the lowest bony prominences are supported by the air cells.
- Make sure that the cushion is properly sized to the individual and the wheelchair.

## Cover Removal and Placement

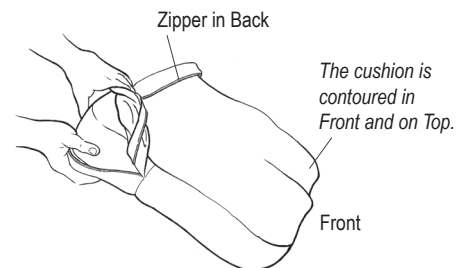
**Warning:** The AirLITE must be used with the provided cover.

**Cover Removal:** Unzip the cover. Remove the cover from the cushion.

**Cover Placement:** Unzip the cover. Insert the cushion into the cover, front edge first, with the contoured foam facing up and the zipper in the back. Close the zipper.

*Note: Refer to the label on the bottom of the cushion and to the image on the cover to correctly orient the cover.*

*Note: Replace any hook fastener strips that were removed prior to washing.*

## Troubleshooting

*For additional assistance, contact the equipment provider, distributor, or Customer Support.*

The cushion feels uncomfortable or unstable.	<ul style="list-style-type: none"> <li>» Confirm that the cushion is properly sized to the individual.</li> <li>» Confirm that the contoured foam faces up and the zipper is in the back.</li> <li>» Confirm that there are no obstructions between the cushion and the seating surface.</li> <li>» Consult with a clinician.</li> </ul> <p>If the cushion still feels uncomfortable or unstable after checking all of the above, immediately consult the prescribing healthcare professional.</p>
The cushion slides on the wheelchair.	<ul style="list-style-type: none"> <li>» Make sure that the non-skid bottom of the cushion cover faces away from the individual.</li> <li>» If a seat insert is used, make sure it is inside the cover, beneath the foam base.</li> <li>» When possible, use hook and loop fasteners to further secure the cushion to the wheelchair. Hook fasteners are available upon request.</li> </ul>
The cushion exhibits unusual stiffness;the cushion has been in temperatures lower than 32°F (0°C)	Before use, allow the cushion to warm to room temperature.
A component is damaged.	Contact Customer Support for a replacement.

## Storage, Transport, Disposal and Recycling

**Storage or Transport:** Clean and disinfect the product. Place in a container that protects the product from moisture, contaminants and damage. After storing or transporting, inspect the product to ensure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

**Disposal:** The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incineration must be performed by a qualified, licensed waste management facility.

**Recycling:** Contact a local recycling agency to determine recycling options for the product.

## Cleaning and Disinfecting

### ⚠ Warnings:

- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. If the product becomes soiled, or between use by different individuals: clean, disinfect, and check the product for proper functioning.
- Clean the product regularly and when it becomes soiled.
- Follow all manufacturer bleach container and germicidal disinfectant safety guidelines.

*Recommendation: When washing by hand, use room-temperature water.*

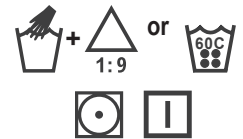
### Cover Care

Before cleaning and disinfecting, remove the cover from the cushion. Remove any hook fastener pieces from the bottom of the cover.

**To Clean the Cover:** Machine wash cold with mild detergent, gentle cycle or wipe clean with neutral detergent and water. Rinse thoroughly. Tumble dry low or air dry.

**To Disinfect the Cover:** Hand wash, using 1 part household liquid bleach per 9 parts water. Keep the product wet with bleach solution for 10 minutes, then rinse thoroughly. Or machine wash in hot water (60°C). Tumble dry low or air dry.

### Cushion Care

#### ⚠ Cautions:

- DO NOT autoclave, steam clean, or expose cushion to temperatures in excess of 150°F (65°C).
- DO NOT submerge the foam in any liquid, including water.
- Use of the following may cause damage to the cushion: caustic, automatic dishwasher detergents; cleaning products that contain petroleum or organic solvents, including acetone, toluene, Methyl Ethyl Ketone (MEK), naphtha, dry cleaning fluid, adhesive remover; ozone gas; ultraviolet light cleaning methods.
- DO NOT machine wash or machine dry the cushion.

### Clean and Disinfect the Cushion

**To Clean the Cushion:** Remove the cover. Wipe gently with a cloth dampened with water and a household detergent. Rinse cloth and re-wipe. Allow foam to air dry thoroughly before putting the cover on.

**⚠ Warning:** Disinfectants are not effective on porous surfaces such as foam. If the foam portion of the product becomes soiled or contaminated, DO NOT use with multiple individuals. Replace soiled or contaminated foam components.

### Care Symbols

Machine wash cold, gentle.	Wipe with a damp cloth.	Tumble dry low.	Line dry.	Hand wash.
Bleach (1 part bleach: 9 parts water).	Machine wash hot, normal, at the temperature shown.	Do not machine wash.	Do not machine dry.	Dry flat.

## Limited Warranty

Limited Warranty Term, 24 months from the date the product was originally purchased. The warranty does not apply to punctures, tears, burns, or misuse. See also the Limited Warranty supplement provided with the product, or contact Customer Support.

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Permobil LTD