


Seat Slings, Back Upholstery and Cushions

Section 5



TENSION ADJUSTABLE BOLT ON SEAT SLING

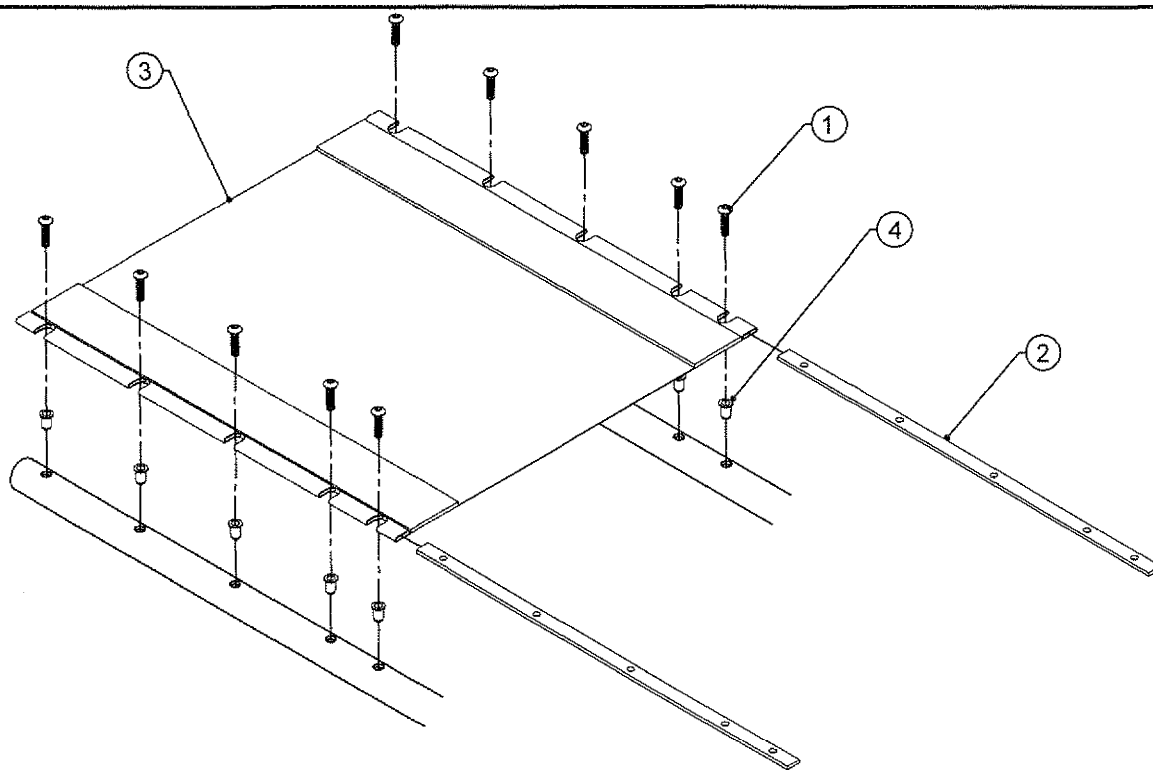

NOTE: ON THE TiLite™ FIXED BACKREST ZR, ZRA, TR (SERIES 2), YR (SERIES 2)
SEE TABLE TWO FOR RAILS AND TABLE SEVEN FOR SEAT SLINGS

ITEM	QTY	PART NO.	DESCRIPTION
1	6-12	HD100071	10-32 X 3/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
2	2	TABLES 1,2, 3	SEAT RAIL
3	1	TABLES 4, 5, 6, 7	SEAT SLING - BLACK SAIL CLOTH (SEE NEXT PAGE)
4	6-12	HD100042	10-32 THREADED INSERT, OPEN END, ZINC, .02-.13 GRIP LENGTH
1,2		TABLES 1,2, 3	BOLT ON SEAT RAIL ASSEMBLY, PAIR

TABLE ONE - ZR, ZRA, TR(SERIES 2), YR(SERIES 2)
Aero Z, Aero R, YG, YGS, X, SX, XC

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
12"	AS100719	CP100081
13"	AS100720	CP100082
14"	AS100721	CP100083
15"	AS100722	CP100084
16"	AS100723	CP100085
17"	AS100724	CP100086
18"	AS100725	CP100091
19"	AS101073	CP100907
20"	AS101074	CP100908

TABLE TWO - FIXED BACKREST ZR, ZRA,
TR (SERIES 2), YR (SERIES 2)

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
12"	AS101072	CP100670
13"	AS100719	CP100081
14"	AS100720	CP100082
15"	AS100721	CP100083
16"	AS100722	CP100084
17"	AS100723	CP100085
18"	AS100724	CP100086
19"	AS100725	CP100091
20"	AS101073	CP100907

TABLE THREE - TR, TRA, EVO, YR, YRA, FIXED BACKREST TR, FIXED BACKREST YR

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
12"	AS101072	CP100670
13"	AS100719	CP100081
14"	AS100720	CP100082
15"	AS100721	CP100083
16"	AS100722	CP100084
17"	AS100723	CP100085
18"	AS100724	CP100086
19"	AS100725	CP100091
20"	AS101073	CP100907

TENSION ADJUSTABLE BOLT ON SEAT SLING (CONT.)

TABLE FOUR - TR, TRA, EVO, YR, YRA, FIXED BACKREST TR, FIXED BACKREST YR

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP101480	CP100020	CP100022	CP100025					
13"	CP101481	CP100021	CP100023	CP100026	CP100029	CP100034			
14"			CP100024	CP100027	CP100030	CP100035			
15"				CP100028	CP100031	CP100036	CP100040		
16"					CP100032	CP100037	CP100041	CP100044	CP101464
17"					CP100033	CP100038	CP100042	CP100045	CP101466
18"						CP100039	CP100043	CP100046	CP101468
19"						CP101470	CP101471	CP101472	CP101473
20"						CP101475	CP101476	CP101477	CP101478

TABLE FIVE - ZR, ZRA, TR(SERIES 2), YR(SERIES 2) Aero Z, Aero R, YG, YGS

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP100020	CP100022	CP100025	CP106793	CP106797	CP106798	CP106801	CP106805	CP106809
13"	CP100021	CP100023	CP100026	CP100029	CP100034	CP106799	CP106802	CP106806	CP106810
14"	CP106734	CP100024	CP100027	CP100030	CP100035	CP106800	CP106803	CP106807	CP106811
15"	CP106735	CP106743	CP100028	CP100031	CP100036	CP100040	CP106804	CP106808	CP106812
16"	CP106736	CP106744	CP106788	CP100032	CP100037	CP100041	CP100044	CP101464	CP101465
17"	CP106737	CP106745	CP106789	CP100033	CP100038	CP100042	CP100045	CP101466	CP101467
18"	CP106738	CP106746	CP106790	CP106794	CP100039	CP100043	CP100046	CP101468	CP101469
19"	CP106739	CP106747	CP106791	CP106795	CP101470	CP101471	CP101472	CP101473	CP101474
20"	CP106740	CP106748	CP106792	CP106796	CP101475	CP101476	CP101477	CP101478	CP101479

TABLE SIX - X, SX, XC

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP100289	CP100291	CP100294						
13"	CP100290	CP100292	CP100295	CP100298	CP100303				
14"		CP100293	CP100296	CP100299	CP100304				
15"			CP100297	CP100300	CP100305	CP100309			
16"				CP100301	CP100306	CP100310	CP100313	CP101351	CP101352
17"				CP100302	CP100307	CP100311	CP100314	CP101353	CP101354
18"					CP100308	CP100312	CP100315	CP101355	CP101356
19"					CP101357	CP101358	CP101359	CP101360	CP101361
20"					CP101362	CP101363	CP101364	CP101365	CP101366


TABLE SEVEN - FIXED BACKREST ZR, ZRA, TR(SERIES 2), YR (SERIES 2)

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP101480	CP100020	CP100022	CP100025	CP106793	CP106797	CP106798	CP106801	CP106805
13"	CP101481	CP100021	CP100023	CP100026	CP100029	CP100034	CP106799	CP106802	CP106806
14"	CP106725	CP106734	CP100024	CP100027	CP100030	CP100035	CP106800	CP106803	CP106807
15"	CP106726	CP106735	CP106743	CP100028	CP100031	CP100036	CP100040	CP106804	CP106808
16"	CP106727	CP106736	CP106744	CP106788	CP100032	CP100037	CP100041	CP100044	CP101464
17"	CP106728	CP106737	CP106745	CP106789	CP100033	CP100038	CP100042	CP100045	CP101466
18"	CP106729	CP106738	CP106746	CP106790	CP106794	CP100039	CP100043	CP100046	CP101468
19"	CP106730	CP106739	CP106747	CP106791	CP106795	CP101470	CP101471	CP101472	CP101473
20"	CP106731	CP106740	CP106748	CP106792	CP106796	CP101475	CP101476	CP101477	CP101478



TENSION ADJUSTABLE BOLT ON SEAT SLING

TiLite™ ZRA (SERIES 2), ZR (SERIES 2), AERO Z (SERIES 2)

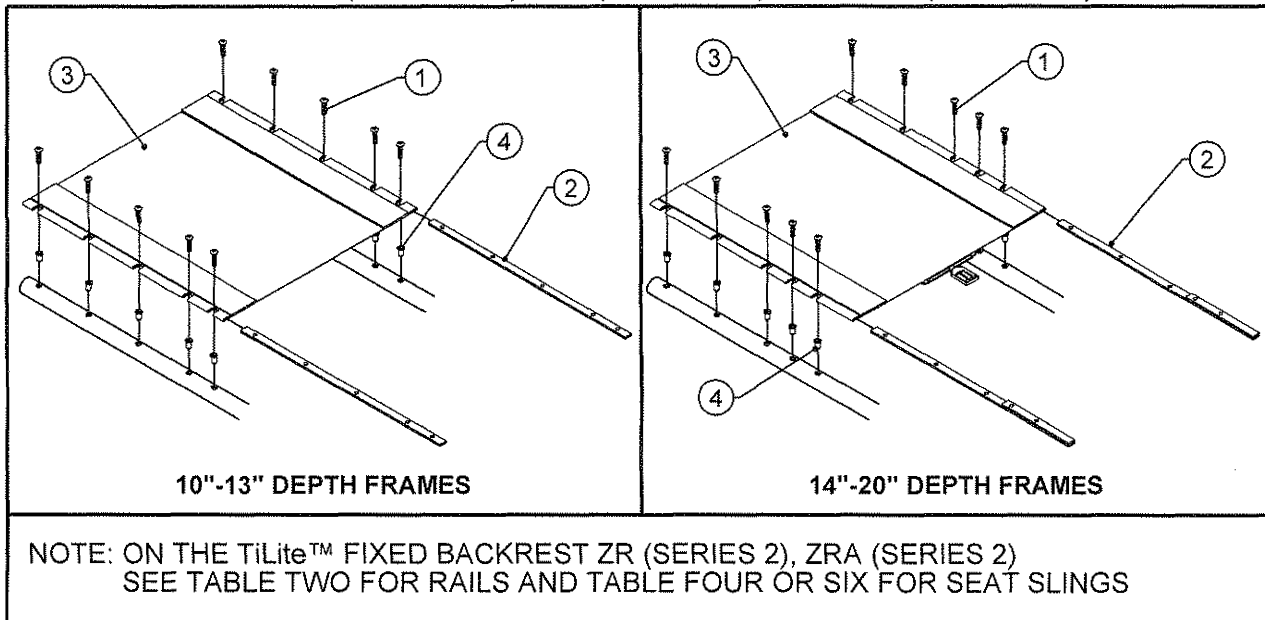

ITEM	QTY	PART NO.	DESCRIPTION
1	6-12	HD100071	10-32 X 3/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
2	2	TABLE ONE AND TWO	SEAT RAIL
3	1	TABLES 3, 4, 5, AND 6	SEAT SLING - BLACK SAIL CLOTH (SEE NEXT PAGE)
4	6-12	HD100142	RIVNUT, 10-32 X .130 - .225 GRIP RANGE
1-2, 4		TABLE ONE AND TWO	BOLT ON SEAT RAIL ASSEMBLY, PAIR

TABLE ONE - FOLDING BACKREST CHAIR SEAT RAILS

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
10"	AS104398	CP101548
11"	AS101069	CP100670
12"	AS100719	CP100081
13"	AS100720	CP100082
14"	AS111118	CP104789
15"	AS111119	CP104790
16"	AS111120	CP104791
17"	AS111121	CP104792
18"	AS111122	CP104793
19"	AS111123	CP104794
20"	AS111124	CP104795

TABLE TWO - FIXED BACKREST ZR (SERIES 2), ZRA (SERIES 2) SEAT RAILS

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
10"	AS116065	CP101598
11"	AS104398	CP101548
12"	AS101069	CP100670
13"	AS100719	CP100081
14"	AS100720	CP100082
15"	AS111118	CP104789
16"	AS111119	CP104790
17"	AS111120	CP104791
18"	AS111121	CP104792
19"	AS111122	CP104793
20"	AS111123	CP104794



Parts Manual

TENSION ADJUSTABLE BOLT ON SEAT SLING

TiLite™ ZRA (SERIES 2), ZR (SERIES 2), AERO Z (SERIES 2)

TABLE THREE - FOLDING BACKREST CHAIR SEAT SLING WITH ISLEEVE

CHAIR WIDTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
14"	CP100024	CP104798	CP104801	CP104806	CP106769	CP106774	CP106780	CP106786
15"	CP106743	CP104799	CP104802	CP104807	CP104813	CP106775	CP106781	CP106787
16"	CP106744	CP106751	CP104803	CP104808	CP104814	CP104819	CP104824	CP104829
17"	CP106745	CP106752	CP104804	CP104809	CP104815	CP104820	CP104825	CP104830
18"	CP106746	CP106753	CP106759	CP104810	CP104816	CP104821	CP104826	CP104831
19"	CP106747	CP106754	CP106760	CP104811	CP104817	CP104822	CP104827	CP104832
20"	CP106748	CP106755	CP106761	CP104812	CP104818	CP104823	CP104828	CP104833

TABLE FOUR - FIXED BACKREST ZR (SERIES 2), ZRA (SERIES 2) SEAT SLING WITH ISLEEVE

CHAIR WIDTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
14"	CP106734	CP107359	CP104798	CP104801	CP104806	CP106769	CP106774	CP106780
15"	CP106735	CP107360	CP104799	CP104802	CP104807	CP104813	CP106775	CP106781
16"	CP106736	CP107361	CP106751	CP104803	CP104808	CP104814	CP104819	CP104824
17"	CP106737	CP107362	CP106752	CP104804	CP104809	CP104815	CP104820	CP104825
18"	CP106738	CP107363	CP106753	CP106759	CP104810	CP104816	CP104821	CP104826
19"	CP106739	CP107364	CP106754	CP106760	CP104811	CP104817	CP104822	CP104827
20"	CP106740	CP107365	CP106755	CP106761	CP104812	CP104818	CP104823	CP104828

TABLE FIVE - FOLDING BACKREST CHAIR SEAT SLING WITH NO ISLEEVE

CHAIR WIDTH	10" SEAT DEPTH	11" SEAT DEPTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
10"	CP103517	CP103512	CP106732	CP106741	CP106749	CP106756	CP106762	CP106765	CP106770	CP106776	CP106782
11"	CP103518	CP103513	CP106733	CP106742	CP106750	CP106757	CP106763	CP106766	CP106771	CP106777	CP106783
12"	CP103519	CP101480	CP100020	CP100022	CP104796	CP106758	CP106764	CP106767	CP106772	CP106778	CP106784
13"	CP103520	CP101481	CP100021	CP100023	CP104797	CP104800	CP104805	CP106768	CP106773	CP106779	CP106785
14"	CP106718	CP106725	CP106734	CP100024	CP107310	CP107317	CP107324	CP107331	CP107338	CP107345	CP107352
15"	CP106719	CP106726	CP106735	CP106743	CP107311	CP107318	CP107325	CP107332	CP107339	CP107346	CP107353
16"	CP106720	CP106727	CP106736	CP106744	CP107312	CP107319	CP107326	CP107333	CP107340	CP107347	CP107354
17"	CP106721	CP106728	CP106737	CP106745	CP107313	CP107320	CP107327	CP107334	CP107341	CP107348	CP107355
18"	CP106722	CP106729	CP106738	CP106746	CP107314	CP107321	CP107328	CP107335	CP107342	CP107349	CP107356
19"	CP106723	CP106730	CP106739	CP106747	CP107315	CP107322	CP107329	CP107336	CP107343	CP107350	CP107357
20"	CP106724	CP106731	CP106740	CP106748	CP107316	CP107323	CP107330	CP107337	CP107344	CP107351	CP107358


TABLE SIX - FIXED BACKREST ZR (SERIES 2), ZRA (SERIES 2) SEAT SLING WITH NO ISLEEVE

CHAIR WIDTH	10" SEAT DEPTH	11" SEAT DEPTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
10"	CP100797	CP103517	CP103512	CP106732	CP106741	CP106749	CP106756	CP106762	CP106765	CP106770	CP106776
11"	CP100797	CP103518	CP103513	CP106733	CP106742	CP106750	CP106757	CP106763	CP106766	CP106771	CP106777
12"	CP100797	CP103519	CP101480	CP100020	CP100022	CP104796	CP106758	CP106764	CP106767	CP106772	CP106778
13"	CP100797	CP103520	CP101481	CP100021	CP100023	CP104797	CP104800	CP104805	CP106768	CP106773	CP106779
14"	CP100797	CP106718	CP106725	CP106734	CP100024	CP107310	CP107317	CP107324	CP107331	CP107338	CP107345
15"	CP100797	CP106719	CP106726	CP106735	CP106743	CP107311	CP107318	CP107325	CP107332	CP107339	CP107346
16"	CP100797	CP106720	CP106727	CP106736	CP106744	CP107312	CP107319	CP107326	CP107333	CP107340	CP107347
17"	CP100797	CP106721	CP106728	CP106737	CP106745	CP107313	CP107320	CP107327	CP107334	CP107341	CP107348
18"	CP100797	CP106722	CP106729	CP106738	CP106746	CP107314	CP107321	CP107328	CP107335	CP107342	CP107349
19"	CP100797	CP106723	CP106730	CP106739	CP106747	CP107315	CP107322	CP107329	CP107336	CP107343	CP107350
20"	CP100797	CP106724	CP106731	CP106740	CP106748	CP107316	CP107323	CP107330	CP107337	CP107344	CP107351



TENSION ADJUSTABLE BOLT ON SEAT SLING

TiLite™ TWIST

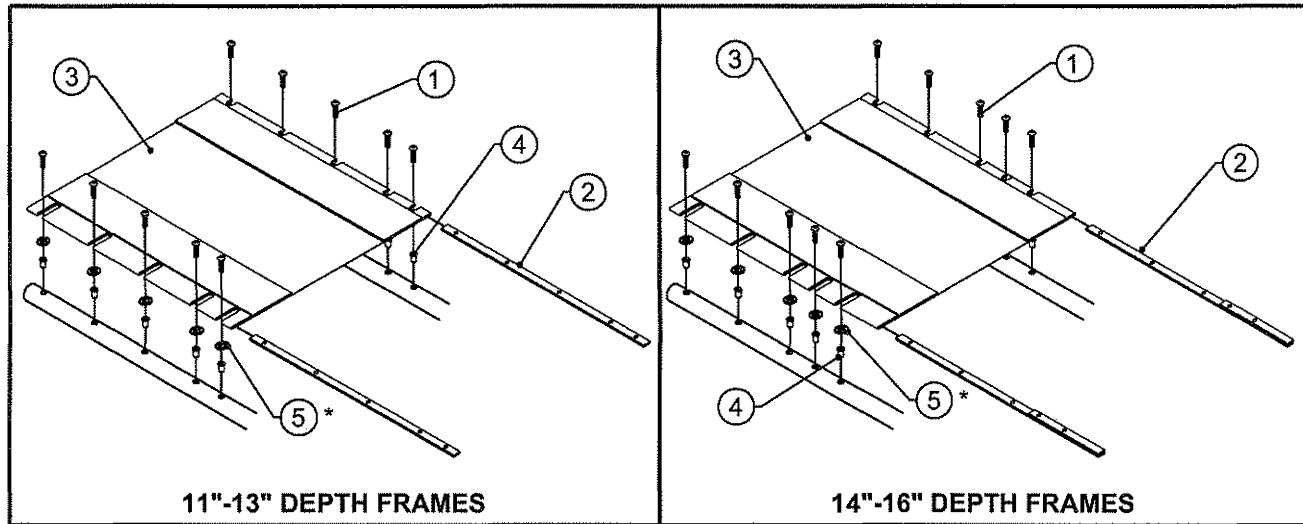

ITEM	QTY	PART NO.	DESCRIPTION
1	6-12	HD100071	10-32 X 3/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
2	2	TABLE ONE	SEAT RAIL
3	1	TABLE TWO	SEAT SLING - BLACK SAIL CLOTH
4	6-10	HD100154	RIVNUT, 10-32 X .130 - .225 GRIP RANGE
5*	5	HD100289	#10 AN WASHER LIGHT SERIES
1,2		TABLE ONE	BOLT ON SEAT RAIL ASSEMBLY, PAIR

TABLE ONE - SEAT RAILS

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
8"-11"	AS111926	CP100670
9"-12"	AS111927	CP100081
10"-13"	AS111928	CP100082
11"-14"	AS111929	CP104789
12"-15"	AS111930	CP104790
13"-16"	AS111931	CP104791

TABLE TWO - SEAT SLING

CHAIR WIDTH	8"-11" SEAT DEPTH	9"-12" SEAT DEPTH	10"-13" SEAT DEPTH	11"-14" SEAT DEPTH	12"-15" SEAT DEPTH	13"-16" SEAT DEPTH
8"-10"	CP105504					
9"-11"	CP105505	CP105506				
10"-12"		CP105507	CP105508			
11"-13"			CP105509	CP105510		
12"-14"				CP105511	CP105512	
13"-15"					CP105513	CP105514


* NOTE: FOR ITEM 5
WASHERS ONLY NEEDED FOR SMALLEST WIDTH CONFIGURATION



Parts Manual

TENSION ADJUSTABLE BOLT ON SEAT SLING

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2)

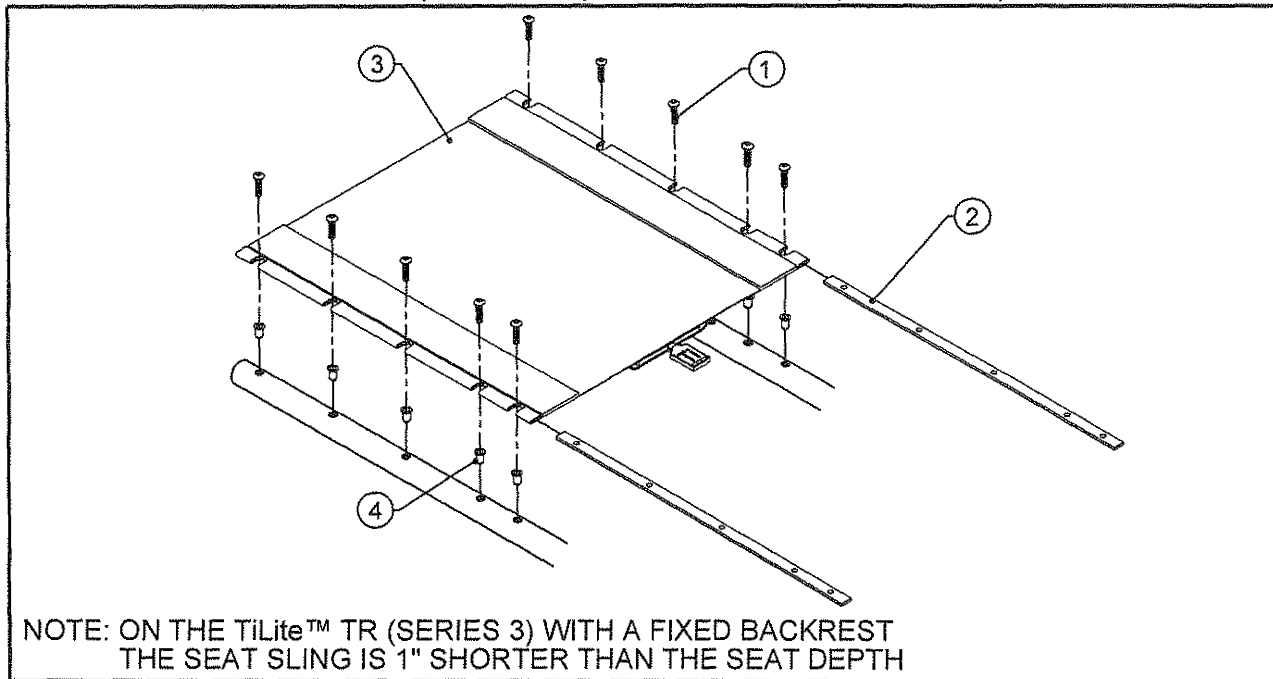

ITEM	QTY	PART NO.	DESCRIPTION
1	6-12	HD100071	10-32 X 3/4" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
2	2	TABLE ONE AND TWO	SEAT RAIL
3	1	TABLES THREE AND FOUR	SEAT SLING - BLACK SAIL CLOTH (SEE NEXT PAGE)
4	6-12	HD100154	RIVNUT, 10-32 X .130 - .225 GRIP RANGE
1,2		TABLE ONE AND TWO	BOLT ON SEAT RAIL ASSEMBLY, PAIR

TABLE ONE - FOLDING BACKREST CHAIRS SEAT RAILS

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
12"	AS100719	CP100081
13"	AS100720	CP100082
14"	AS100721	CP100083
15"	AS100722	CP100084
16"	AS100723	CP100085
17"	AS100724	CP100086
18"	AS100725	CP100091
19"	AS101073	CP100907
20"	AS101074	CP100908

TABLE TWO - FIXED BACKREST TR (SERIES 3) SEAT RAILS

CHAIR DEPTH	SEAT RAIL ASSEMBLY	SEAT RAIL
12"	AS101072	CP100670
13"	AS100719	CP100081
14"	AS100720	CP100082
15"	AS100721	CP100083
16"	AS100722	CP100084
17"	AS100723	CP100085
18"	AS100724	CP100086
19"	AS100725	CP100091
20"	AS101073	CP100907



Parts Manual

TENSION ADJUSTABLE BOLT ON SEAT SLING

TiLite™ TR (SERIES 3), AERO T, TRA (SERIES 2)

TABLE THREE - FOLDING BACKREST CHAIR SEAT SLING WITH ISLEEVE

CHAIR WIDTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
14"	CP100024	CP106904	CP106911	CP106918	CP106925	CP106932	CP106939	CP106946
15"	CP106743	CP106905	CP106912	CP106919	CP106926	CP106933	CP106940	CP106947
16"	CP106744	CP106906	CP106913	CP106920	CP106927	CP106934	CP106941	CP106948
17"	CP106745	CP106907	CP106914	CP106921	CP106928	CP106935	CP106942	CP106949
18"	CP106746	CP106908	CP106915	CP106922	CP106929	CP106936	CP106943	CP106950
19"	CP106747	CP106909	CP106916	CP106923	CP106930	CP106937	CP106944	CP106951
20"	CP106748	CP106910	CP106917	CP106924	CP106931	CP106938	CP106945	CP106952

TABLE FOUR - FIXED BACKREST TR (SERIES 3) SEAT SLING WITH ISLEEVE

CHAIR WIDTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
14"	CP106734	CP107359	CP106904	CP106911	CP106918	CP106925	CP106932	CP106939
15"	CP106735	CP107360	CP106905	CP106912	CP106919	CP106926	CP106933	CP106940
16"	CP106736	CP107361	CP106906	CP106913	CP106920	CP106927	CP106934	CP106941
17"	CP106737	CP107362	CP106907	CP106914	CP106921	CP106928	CP106935	CP106942
18"	CP106738	CP107363	CP106908	CP106915	CP106922	CP106929	CP106936	CP106943
19"	CP106739	CP107364	CP106909	CP106916	CP106923	CP106930	CP106937	CP106944
20"	CP106740	CP107365	CP106910	CP106917	CP106924	CP106931	CP106938	CP106945

TABLE FIVE - FOLDING BACKREST CHAIR SEAT SLING WITH NO ISLEEVE

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP100020	CP100022	CP100025	CP106793	CP106797	CP106798	CP106801	CP106805	CP106809
13"	CP100021	CP100023	CP100026	CP100029	CP100034	CP106799	CP106802	CP106806	CP106810
14"	CP106734	CP100024	CP100027	CP100030	CP100035	CP106800	CP106803	CP106807	CP106811
15"	CP106735	CP106743	CP100028	CP100031	CP100036	CP100040	CP106804	CP106808	CP106812
16"	CP106736	CP106744	CP106788	CP100032	CP100037	CP100041	CP100044	CP101464	CP101465
17"	CP106737	CP106745	CP106789	CP100033	CP100038	CP100042	CP100045	CP101466	CP101467
18"	CP106738	CP106746	CP106790	CP106794	CP100039	CP100043	CP100046	CP101468	CP101469
19"	CP106739	CP106747	CP106791	CP106795	CP101470	CP101471	CP101472	CP101473	CP101474
20"	CP106740	CP106748	CP106792	CP106796	CP101475	CP101476	CP101477	CP101478	CP101479


TABLE FIVE - FIXED BACKREST TR (SERIES 3) CHAIR SEAT SLING WITH NO ISLEEVE

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP101480	CP100020	CP100022	CP100025	CP106793	CP106797	CP106798	CP106801	CP106805
13"	CP101481	CP100021	CP100023	CP100026	CP100029	CP100034	CP106799	CP106802	CP106806
14"	CP106725	CP106734	CP100024	CP100027	CP100030	CP100035	CP106800	CP106803	CP106807
15"	CP106726	CP106735	CP106743	CP100028	CP100031	CP100036	CP100040	CP106804	CP106808
16"	CP106727	CP106736	CP106744	CP106788	CP100032	CP100037	CP100041	CP100044	CP101464
17"	CP106728	CP106737	CP106745	CP106789	CP100033	CP100038	CP100042	CP100045	CP101466
18"	CP106729	CP106738	CP106746	CP106790	CP106794	CP100039	CP100043	CP100046	CP101468
19"	CP106730	CP106739	CP106747	CP106791	CP106795	CP101470	CP101471	CP101472	CP101473
20"	CP106731	CP106740	CP106748	CP106792	CP106796	CP101475	CP101476	CP101477	CP101478



TENSION ADJUSTABLE BY STRAP SEAT SLING

TiLite™ TR, TRC, YR, BB

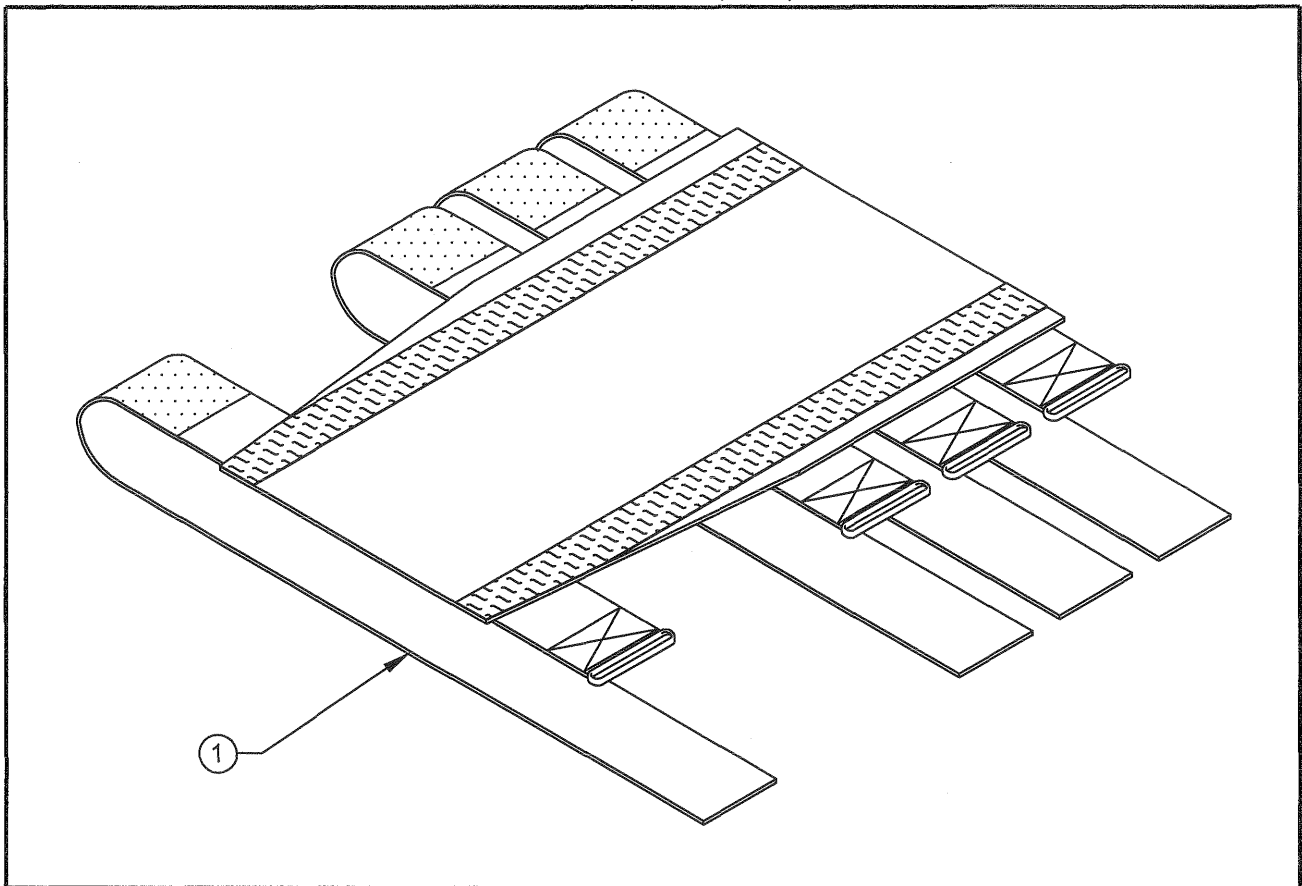


ITEM	QUANTITY	PART NO.	DESCRIPTION
CON 1	1	CP100796	SEAT SLING W/STRAPS CUSTOM

CON CONSULT WITH FACTORY TO SPECIFY SIZE OF SEAT SLING NEEDED
THE SERIAL NUMBER OR ORDER NUMBER OF THE CHAIR WILL BE NEEDED

TENSION ADJUSTABLE BY STRAP SEAT SLING

TiLite™ ZR, ZRA, Z10, AERO Z, TR (SERIES 2), YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2)

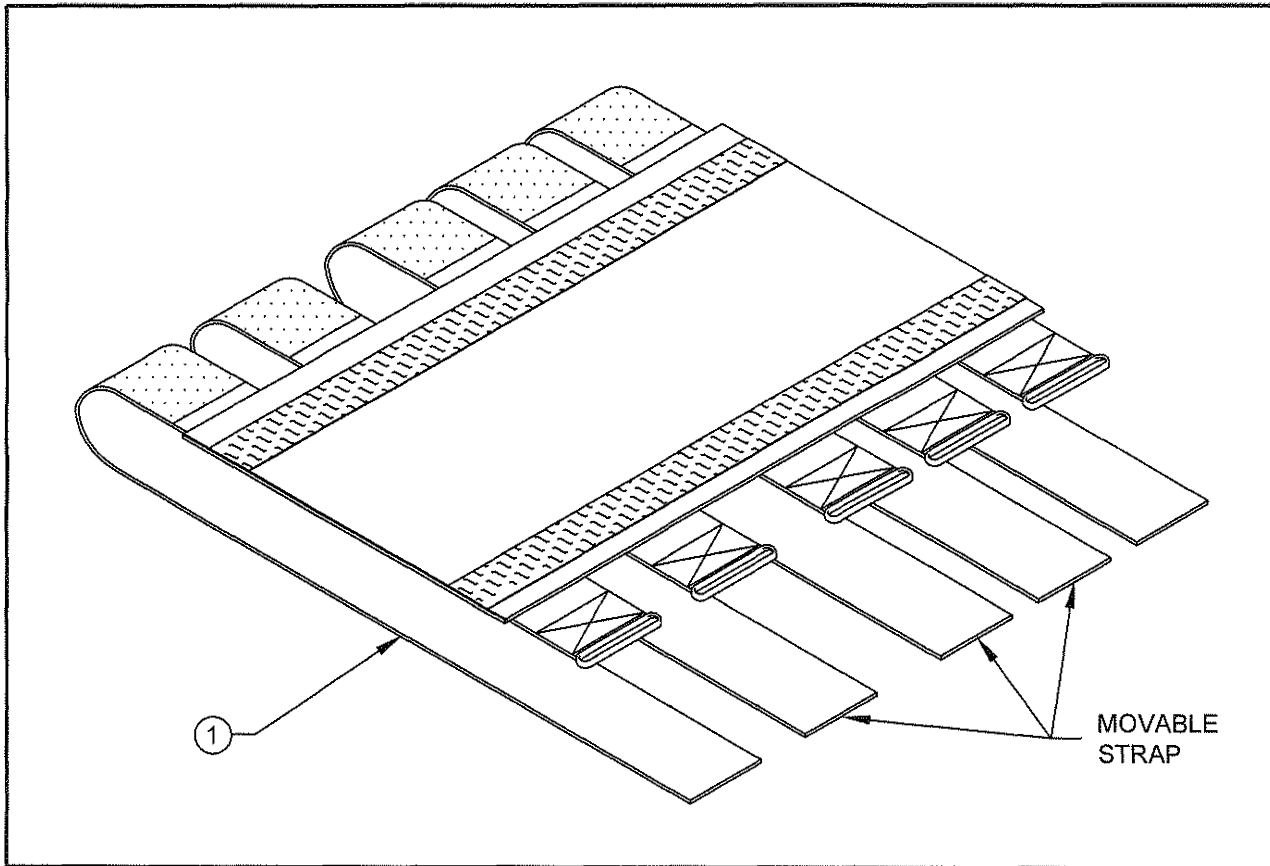


ITEM	QUANTITY	PART NO.	DESCRIPTION
CON 1	1	CP102098	SEAT SLING W/STRAPS CUSTOM

CON CONSULT WITH FACTORY TO SPECIFY SIZE OF SEAT SLING NEEDED
THE SERIAL NUMBER OR ORDER NUMBER OF THE CHAIR WILL BE NEEDED

TENSION ADJ. SLIDE IN SEAT SLING

TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

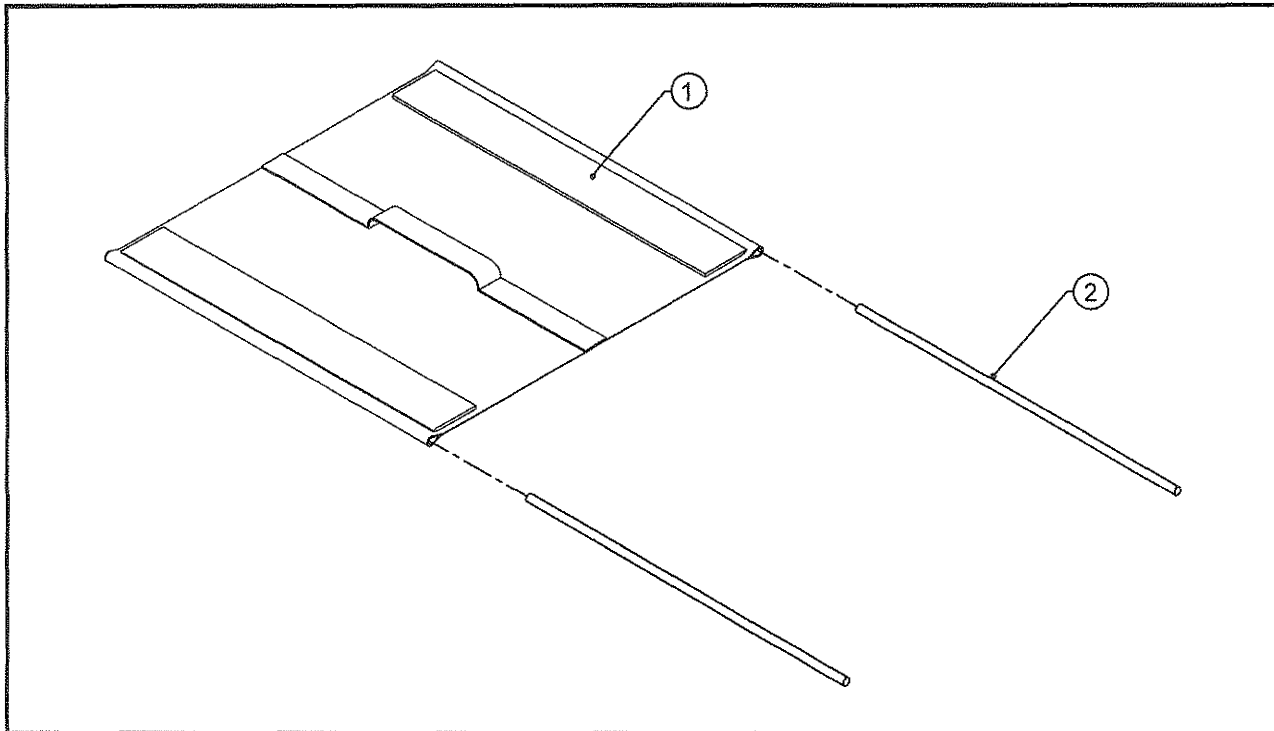

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	SLIDE IN SEAT SLING
2	2	TABLE TWO	SLIDE IN SEAT SLING ROD
1,2		TABLE THREE	SLIDE IN SEAT SLING ASSEMBLY

TABLE ONE - SLIDE IN SEAT SLING

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP102628	CP102629	CP106666	CP106667	CP106668	CP106669	CP106670	CP106671	CP106672
13"	CP102631	CP102632	CP102633	CP102634	CP106673	CP106674	CP106675	CP106676	CP106677
14"	CP102635	CP102636	CP102637	CP102638	CP106678	CP106679	CP106680	CP106681	CP106682
15"	CP106683	CP102639	CP102640	CP102641	CP102642	CP106684	CP106685	CP106686	CP106687
16"	CP106688	CP106689	CP102643	CP102644	CP102645	CP102646	CP102647	CP102648	CP103475
17"	CP106690	CP106691	CP102649	CP102650	CP102651	CP102652	CP102653	CP102654	CP103476
18"	CP106692	CP106693	CP106694	CP102655	CP102656	CP102657	CP102658	CP102659	CP103477
19"	CP106695	CP106696	CP106697	CP102660	CP102661	CP102662	CP102663	CP102664	CP103478
20"	CP106698	CP106699	CP106700	CP102665	CP102666	CP102667	CP102668	CP102669	CP103479
21"	CP106701	CP106702	CP106703	CP106704	CP104587	CP104588	CP104589	CP104590	CP104591
22"	CP106705	CP106706	CP106707	CP106708	CP104592	CP104593	CP104594	CP104595	CP104596

TABLE TWO - SLIDE IN SEAT SLING ROD

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
ALL	CP102686	CP102687	CP102688	CP102689	CP102690	CP102691	CP102692	CP102693	CP102694


TABLE THREE - SLIDE IN SEAT SLING ASSEMBLY

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	AS102988	AS102989	AS115247	AS115253	AS115256	AS115259	AS115263	AS115267	AS115271
13"	AS102991	AS102992	AS102993	AS102994	AS115257	AS115260	AS115264	AS115268	AS115272
14"	AS102995	AS102996	AS102997	AS102998	AS115258	AS115261	AS115265	AS115269	AS115273
15"	AS115232	AS102999	AS103000	AS103001	AS103002	AS115262	AS115266	AS115270	AS115274
16"	AS115233	AS115240	AS103003	AS103004	AS103005	AS103006	AS103007	AS103008	AS103779
17"	AS115234	AS115241	AS103009	AS103010	AS103011	AS103012	AS103013	AS103014	AS103780
18"	AS115235	AS115242	AS115248	AS103015	AS103016	AS103017	AS103018	AS103019	AS103781
19"	AS115236	AS115243	AS115249	AS103020	AS103021	AS103022	AS103023	AS103024	AS103782
20"	AS115237	AS115244	AS115250	AS103025	AS103026	AS103027	AS103028	AS103029	AS103783
21"	AS115238	AS115245	AS115251	AS115254	AS110475	AS110476	AS110477	AS110478	AS110479
22"	AS115239	AS115246	AS115252	AS115255	AS110480	AS110481	AS110482	AS110483	AS110484



TENSION ADJ. SLIDE IN SEAT SLING

TiLite™ TX

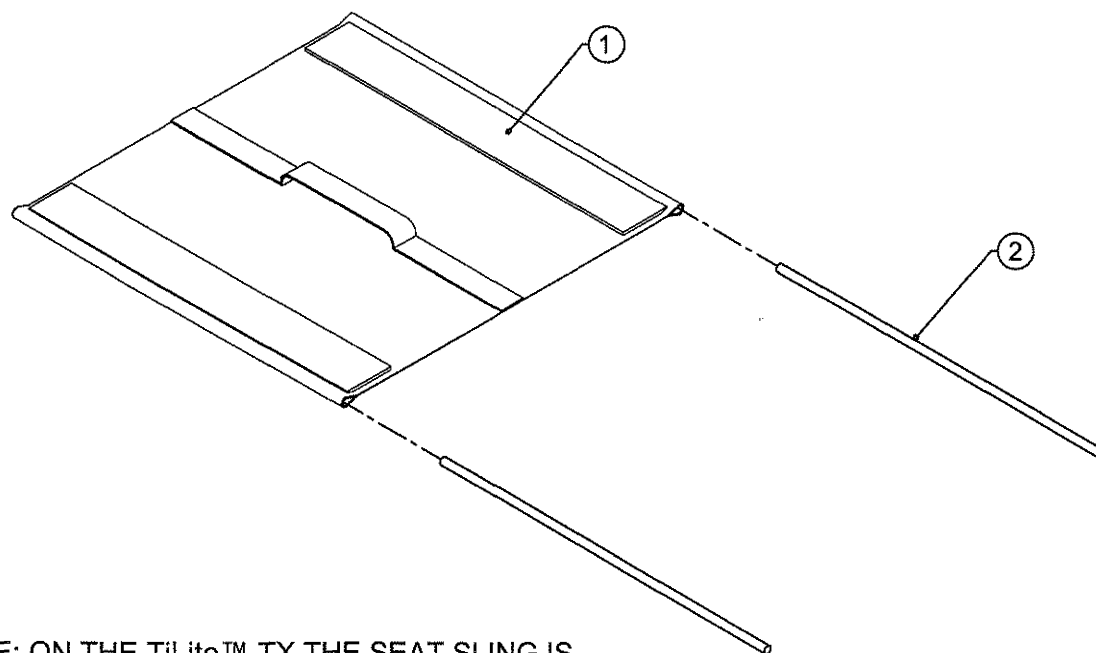

NOTE: ON THE TiLite™ TX THE SEAT SLING IS
1" SHORTER THAN THE SEAT DEPTH

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	SLIDE IN SEAT SLING
2	2	TABLE TWO	SLIDE IN SEAT SLING ROD
1,2		TABLE THREE	SLIDE IN SEAT SLING ASSEMBLY

TABLE ONE - SLIDE IN SEAT SLING

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP102627	CP102628	CP102629	CP106666	CP106667	CP106668	CP106669	CP106670	CP106671
13"	CP102630	CP102631	CP102632	CP102633	CP102634	CP106673	CP106674	CP106675	CP106676
14"	CP106709	CP102635	CP102636	CP102637	CP102638	CP106678	CP106679	CP106680	CP106681
15"	CP106710	CP106683	CP102639	CP102640	CP102641	CP102642	CP106684	CP106685	CP106686
16"	CP106711	CP106688	CP106689	CP102643	CP102644	CP102645	CP102646	CP102647	CP102648
17"	CP106712	CP106690	CP106691	CP102649	CP102650	CP102651	CP102652	CP102653	CP102654
18"	CP106713	CP106692	CP106693	CP106694	CP102655	CP102656	CP102657	CP102658	CP102659
19"	CP106714	CP106695	CP106696	CP106697	CP102660	CP102661	CP102662	CP102663	CP102664
20"	CP106715	CP106698	CP106699	CP106700	CP102665	CP102666	CP102667	CP102668	CP102669

TABLE TWO - SLIDE IN SEAT SLING ROD

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
ALL	CP102685	CP102686	CP102687	CP102688	CP102689	CP102690	CP102691	CP102692	CP102693


TABLE THREE - SLIDE IN SEAT SLING ASSEMBLY

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	AS102987	AS102988	AS102989	AS115247	AS115253	AS115256	AS115259	AS115263	AS115267
13"	AS102990	AS102991	AS102992	AS102993	AS102994	AS115257	AS115260	AS115264	AS115268
14"	AS115283	AS102995	AS102996	AS102997	AS102998	AS115258	AS115261	AS115265	AS115269
15"	AS115284	AS115232	AS102999	AS103000	AS103001	AS103002	AS115262	AS115266	AS115270
16"	AS115285	AS115233	AS115240	AS103003	AS103004	AS103005	AS103006	AS103007	AS103008
17"	AS115286	AS115234	AS115241	AS103009	AS103010	AS103011	AS103012	AS103013	AS103014
18"	AS115287	AS115235	AS115242	AS115248	AS103015	AS103016	AS103017	AS103018	AS103019
19"	AS115288	AS115236	AS115243	AS115249	AS103020	AS103021	AS103022	AS103023	AS103024
20"	AS115289	AS115237	AS115244	AS115250	AS103025	AS103026	AS103027	AS103028	AS103029



SOLID SEAT PAN, ALUMINUM


TiLite™ ZR, ZRA, TR (SERIES 2), YR (SERIES 2), TRA, TR, YR,
EVO, Aero Z, Aero R, TWIST

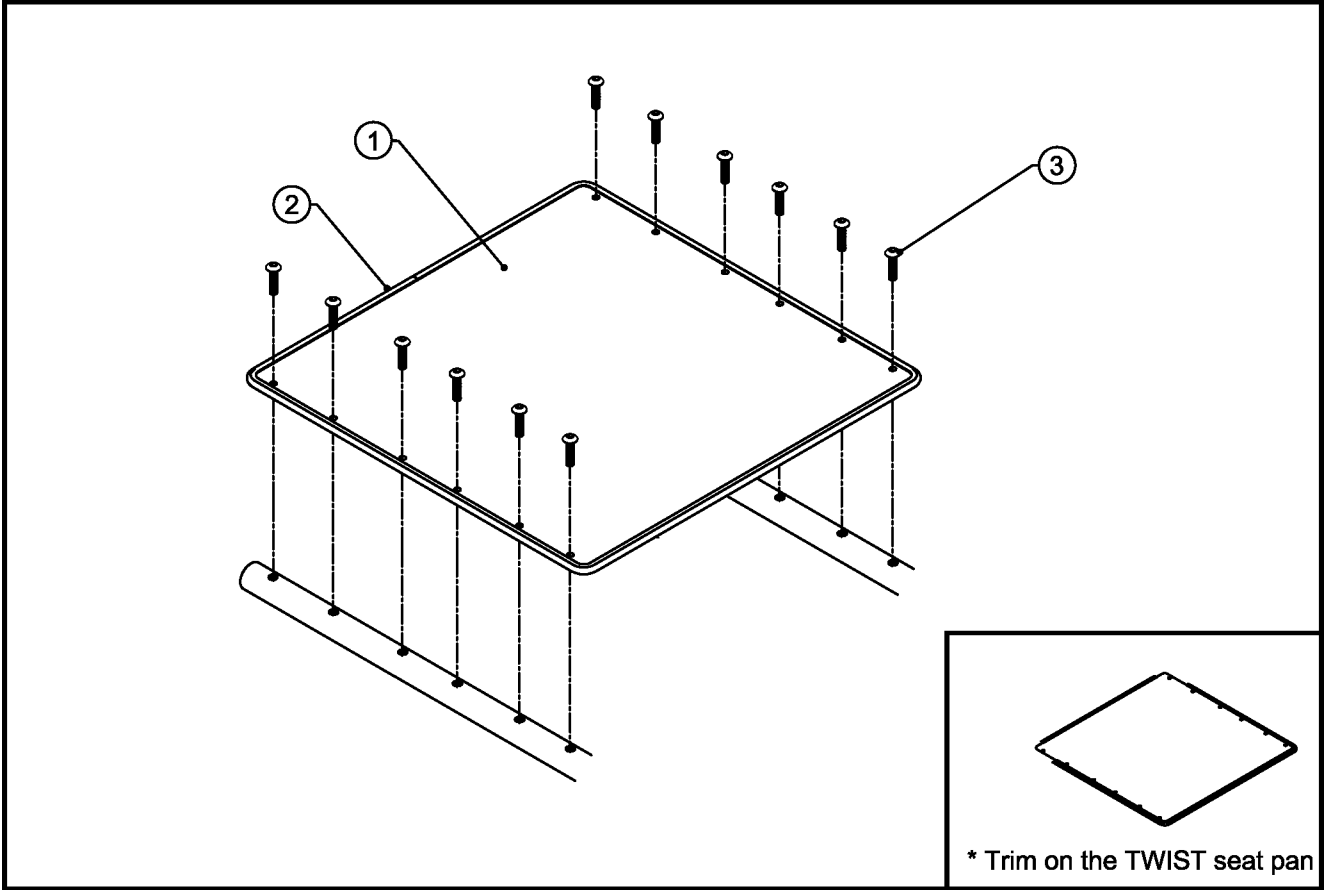

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP102510	ALUMINUM SEAT PAN (CUSTOM SIZED)
2	1	RW100090	EDGE TRIM FOR 1/16" MATERIAL
3	6-12	HD100532	10-32 X 3/4" BUTTON HEAD CAP SCREW, BLACK ZINC
1-3		AS102926	ALUMINUM SEAT PAN

SOLID SEATPAN

TiLite™ Z10

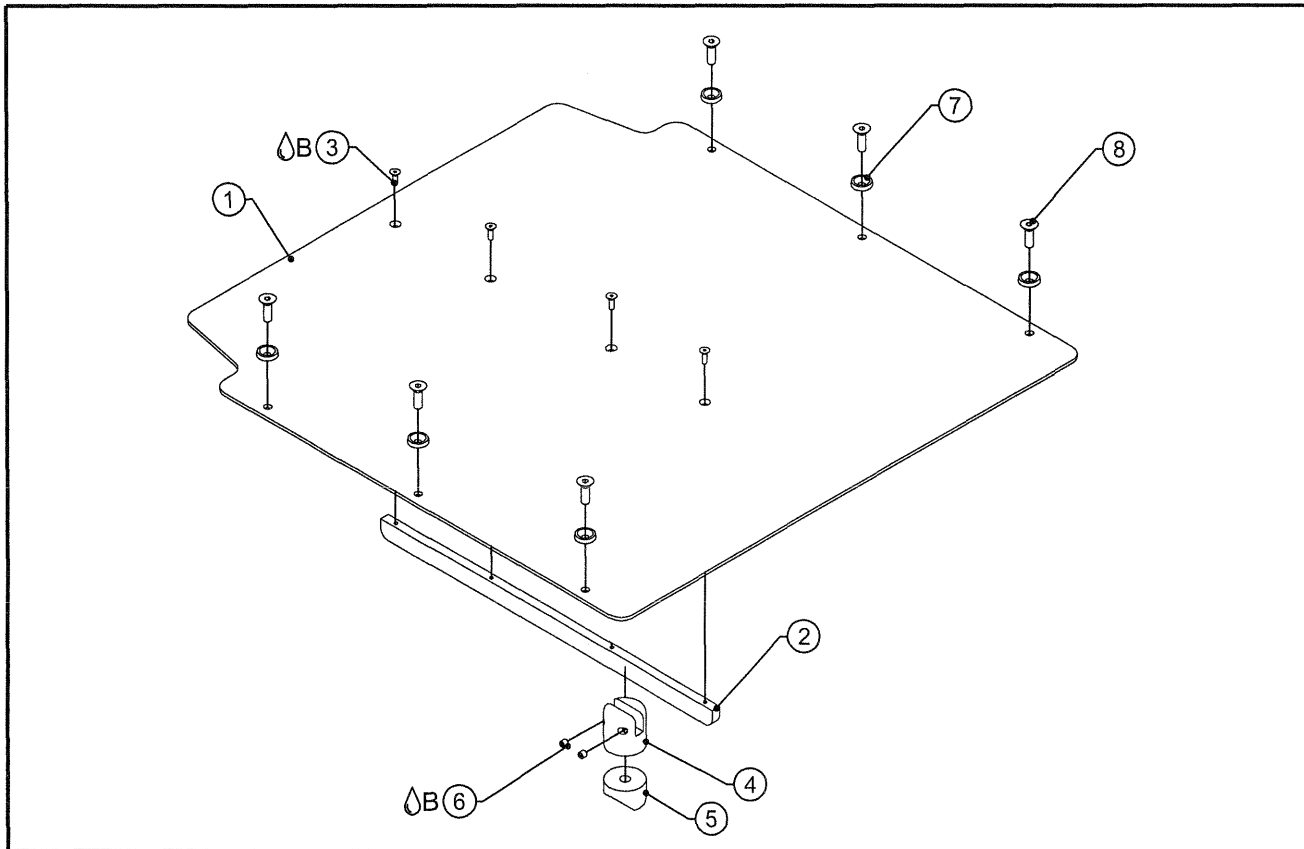


ITEM	QUANTITY	PART NO.	DESCRIPTION
CON 1	1	CP103750	COMPOSITE SEAT PAN (CUSTOM SIZED)
2	1	CP103751	COMPOSITE SEAT PAN RIB
3 ΔB	4	HD100522	4-40 X 3/8" FLAT HEAD CAP SCREW, BLACK ZINC
4	1	CP103777	SEAT PAN SADDLE
5	1	HD100099	COVED WASHER 1" X 1/4" I.D. X 1/4" THK
6 ΔB	2	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
7	6	HD100521	NYLON COUNTERSUNK WASHER, .509 OD X .114 THK
8	6	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
1-8		AS104324	COMPOSITE SOLID SEAT PAN

CON CONSULT WITH FACTORY TO SPECIFY SIZE OF SEATPAN NEEDED
THE SERIAL NUMBER OR ORDER NUMBER OF THE CHAIR WILL BE NEEDED

SOLID SEATPAN

TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2),
TR (SERIES 3), AERO T, TRA (SERIES 2)

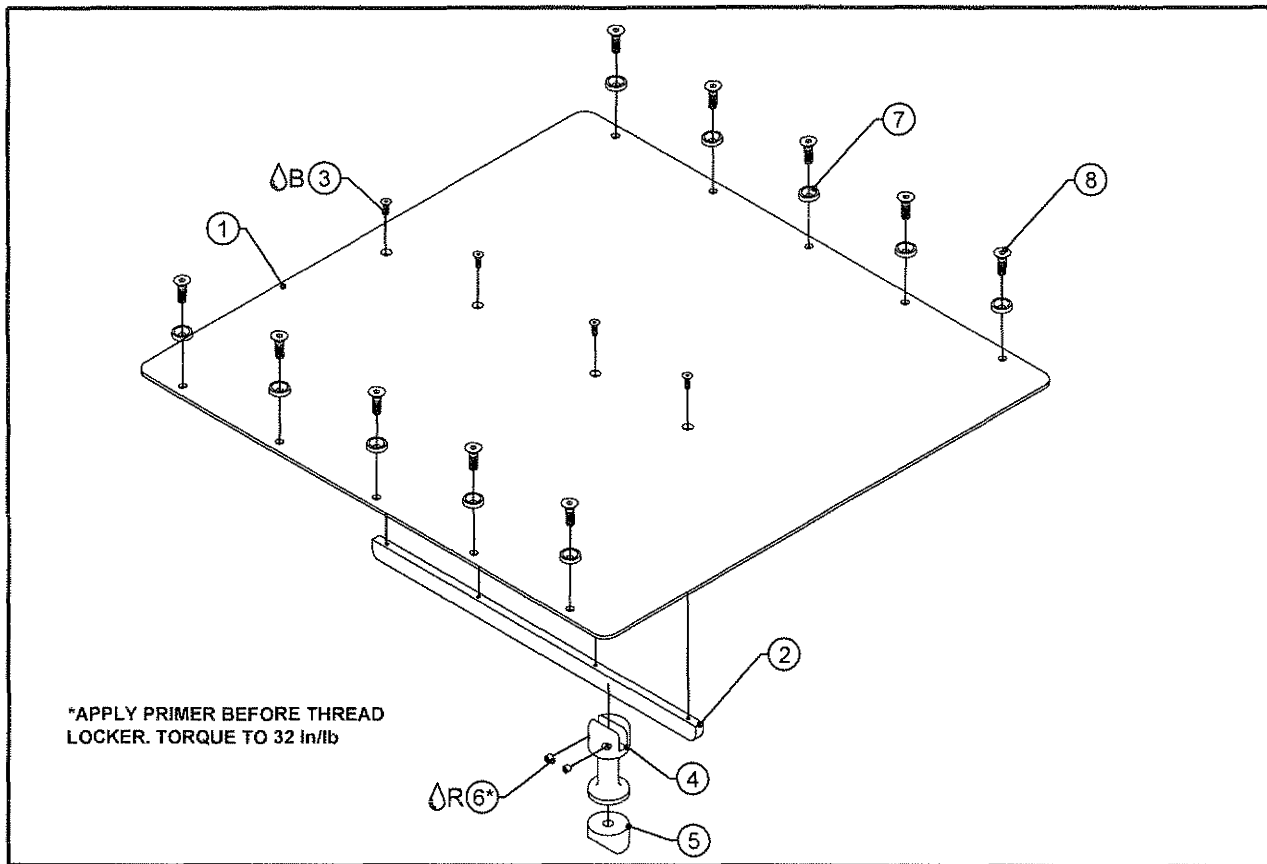


ITEM	QUANTITY	PART NO.	DESCRIPTION
CON 1	1	CP104722	COMPOSITE SEAT PAN (CUSTOM SIZED)
2	1	CP103751	COMPOSITE SEAT PAN RIB
3 ΔB	4	HD100522	4-40 X 3/8" FLAT HEAD CAP SCREW, BLACK ZINC
4A	1	CP104719	SEAT PAN SADDLE FOR 1.25" FRAME
4B	1	CP106962	SEAT PAN SADDLE FOR 1" FRAME
5A	1	HD100099	COVED WASHER 1" X 1/4" I.D. X 1/4" THK
5B	1	HD100014	COVED WASHER 1" OD X 1/4" ID X 1/8" THK
6 ΔR	2	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
7	10	HD100521	NYLON COUNTERSUNK WASHER, .509 OD X .114 THK
8	10	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
1-3,4A,5A-8		AS110923	COMPOSITE SOLID SEAT PAN FOR 1.25" FRAME
1-3,4B,5B-8		AS115291	COMPOSITE SOLID SEAT PAN FOR 1" FRAME

CON CONSULT WITH FACTORY TO SPECIFY SIZE OF SEATPAN NEEDED
THE SERIAL NUMBER OR ORDER NUMBER OF THE CHAIR WILL BE NEEDED

SOLID SEAT PAN, ALUMINUM

TiLite™ 2GX, Aero X, TX, X, SX, Aero X (SERIES 2), 2GX (SERIES 2)

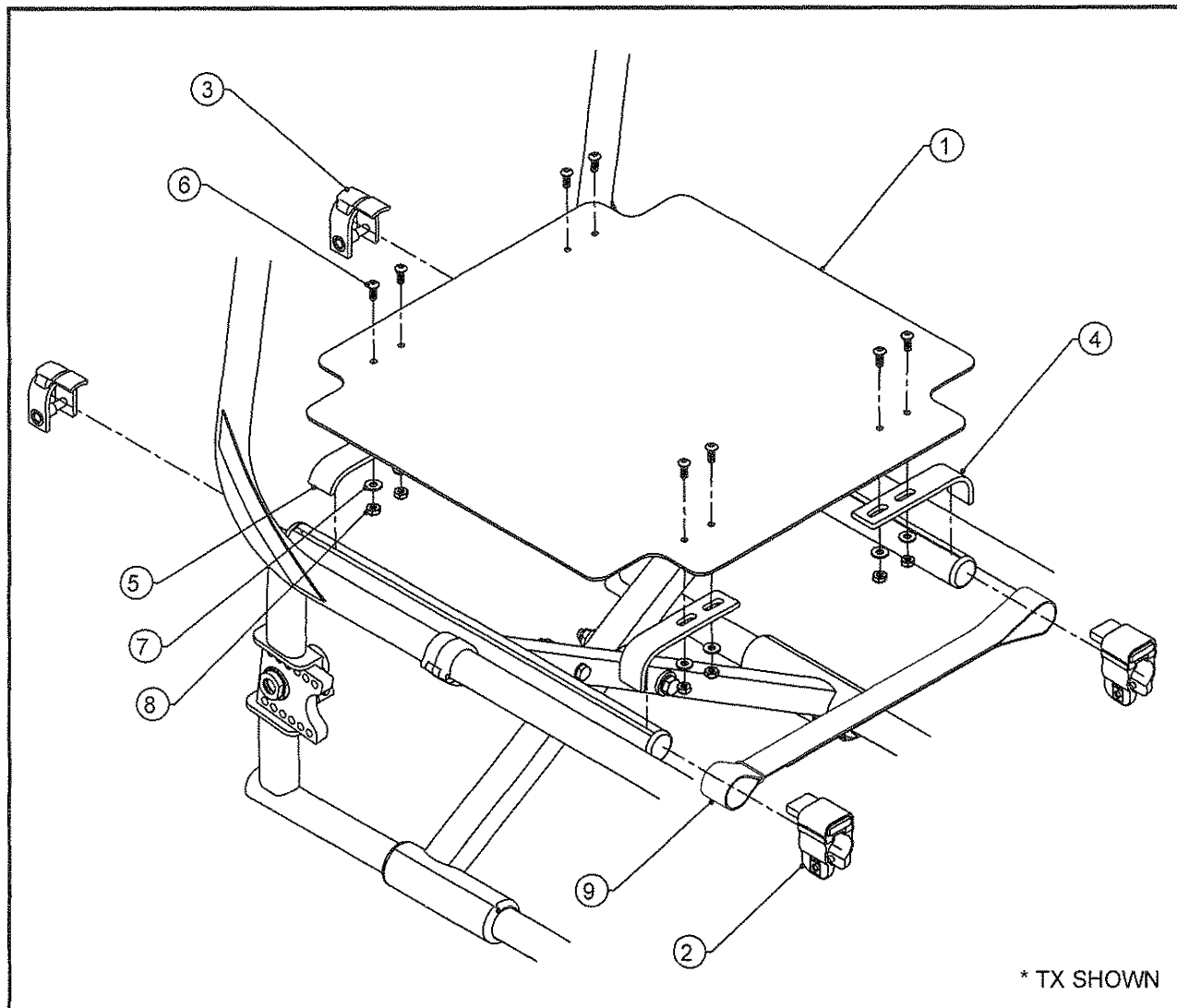

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	TABLE ONE	SOLID SEAT PAN FOR 2GX, Aero X, X, SX (SEE NEXT PAGE)
1B	1	TABLE TWO	SOLID SEAT PAN FOR TX (SEE NEXT PAGE)
2	2	CP103789	FRONT FRAME CLIP FOR SOLID SEAT PAN
3	2	CP103790	REAR FRAME CLIP FOR SOLID SEAT PAN
4	2	CP103791	FRONT FLAT HOOK FOR SOLID SEAT PAN
5	2	CP103792	REAR FLAT HOOK FOR SOLID SEAT PAN
6	8	HD100527	10-32 X 1/2" BUTTON HEAD CAP SCREW, BLACK ZINC
7	8	HD100288	#10 SAE FLAT WASHER
8	8	HD100002	10-32 LOCKNUT NYLOCK
9	1	TABLE THREE	1" STRAP (SEE NEXT PAGE)
1-9		TABLE FOUR	SOLID SEAT PAN ASSEMBLY FOR 2GX, Aero X, X, SX (SEE NEXT PAGE)
1-9		TABLE FIVE	SOLID SEAT PAN ASSEMBLY FOR TX (SEE NEXT PAGE)

SOLID SEAT PAN, ALUMINUM (CONT.)

TiLite™ 2GX, Aero X, TX, X, SX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE ONE - SOLID SEAT PAN FOR 2GX, Aero X, X, SX

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP103796	CP103799	N/A	N/A	N/A	N/A	N/A	N/A	N/A
13"	CP103797	CP103800	CP103803	CP103808	N/A	N/A	N/A	N/A	N/A
14"	CP103798	CP103801	CP103804	CP103809	N/A	N/A	N/A	N/A	N/A
15"	N/A	CP103802	CP103805	CP103810	CP103816	N/A	N/A	N/A	N/A
16"	N/A	N/A	CP103806	CP103811	CP103817	CP103822	CP103827	CP103832	CP103837
17"	N/A	N/A	CP103807	CP103812	CP103818	CP103823	CP103828	CP103833	CP103838
18"	N/A	N/A	N/A	CP103813	CP103819	CP103824	CP103829	CP103834	CP103839
19"	N/A	N/A	N/A	CP103814	CP103820	CP103825	CP103830	CP103835	CP103840
20"	N/A	N/A	N/A	CP103815	CP103821	CP103826	CP103831	CP103836	CP103841

TABLE TWO - SOLID SEAT PAN FOR TX

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	CP103794	CP103796	CP103799	N/A	N/A	N/A	N/A	N/A	N/A
13"	CP103795	CP103797	CP103800	CP103803	CP103808	N/A	N/A	N/A	N/A
14"	N/A	CP103798	CP103801	CP103804	CP103809	N/A	N/A	N/A	N/A
15"	N/A	N/A	CP103802	CP103805	CP103810	CP103816	N/A	N/A	N/A
16"	N/A	N/A	N/A	CP103806	CP103811	CP103817	CP103822	CP103827	CP103832
17"	N/A	N/A	N/A	CP103807	CP103812	CP103818	CP103823	CP103828	CP103833
18"	N/A	N/A	N/A	N/A	CP103813	CP103819	CP103824	CP103829	CP103834
19"	N/A	N/A	N/A	N/A	CP103814	CP103820	CP103825	CP103830	CP103835
20"	N/A	N/A	N/A	N/A	CP103815	CP103821	CP103826	CP103831	CP103836

TABLE THREE - 1" STRAPS

CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
PART NO.	CP102246	CP101517	CP101518	CP101519	CP101520

TABLE FOUR - SOLID SEAT PAN ASSEMBLY FOR 2GX, Aero X, X, SX

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	AS104201	AS104204	N/A	N/A	N/A	N/A	N/A	N/A	N/A
13"	AS104202	AS104205	AS104208	AS104213	N/A	N/A	N/A	N/A	N/A
14"	AS104203	AS104206	AS104209	AS104214	N/A	N/A	N/A	N/A	N/A
15"	N/A	AS104207	AS104210	AS104215	AS104221	N/A	N/A	N/A	N/A
16"	N/A	N/A	AS104211	AS104216	AS104222	AS104227	AS104232	AS104237	AS104242
17"	N/A	N/A	AS104212	AS104217	AS104223	AS104228	AS104233	AS104238	AS104243
18"	N/A	N/A	N/A	AS104218	AS104224	AS104229	AS104234	AS104239	AS104244
19"	N/A	N/A	N/A	AS104219	AS104225	AS104230	AS104235	AS104240	AS104245
20"	N/A	N/A	N/A	AS104220	AS104226	AS104231	AS104236	AS104241	AS104246


TABLE FIVE - SOLID SEAT PAN ASSEMBLY FOR TX

CHAIR WIDTH	12" SEAT DEPTH	13" SEAT DEPTH	14" SEAT DEPTH	15" SEAT DEPTH	16" SEAT DEPTH	17" SEAT DEPTH	18" SEAT DEPTH	19" SEAT DEPTH	20" SEAT DEPTH
12"	AS104199	AS104201	AS104204	N/A	N/A	N/A	N/A	N/A	N/A
13"	AS104200	AS104202	AS104205	AS104208	AS104213	N/A	N/A	N/A	N/A
14"	N/A	AS104203	AS104206	AS104209	AS104214	N/A	N/A	N/A	N/A
15"	N/A	N/A	AS104207	AS104210	AS104215	AS104221	N/A	N/A	N/A
16"	N/A	N/A	N/A	AS104211	AS104216	AS104222	AS104227	AS104232	AS104237
17"	N/A	N/A	N/A	AS104212	AS104217	AS104223	AS104228	AS104233	AS104238
18"	N/A	N/A	N/A	N/A	AS104218	AS104224	AS104229	AS104234	AS104239
19"	N/A	N/A	N/A	N/A	AS104219	AS104225	AS104230	AS104235	AS104240
20"	N/A	N/A	N/A	N/A	AS104220	AS104226	AS104231	AS104236	AS104241



BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ ZR, ZRA, Z10, AERO Z
TR(SERIES 2), YR(SERIES 2), AERO R, ZR(SERIES 2), ZRA(SERIES 2)

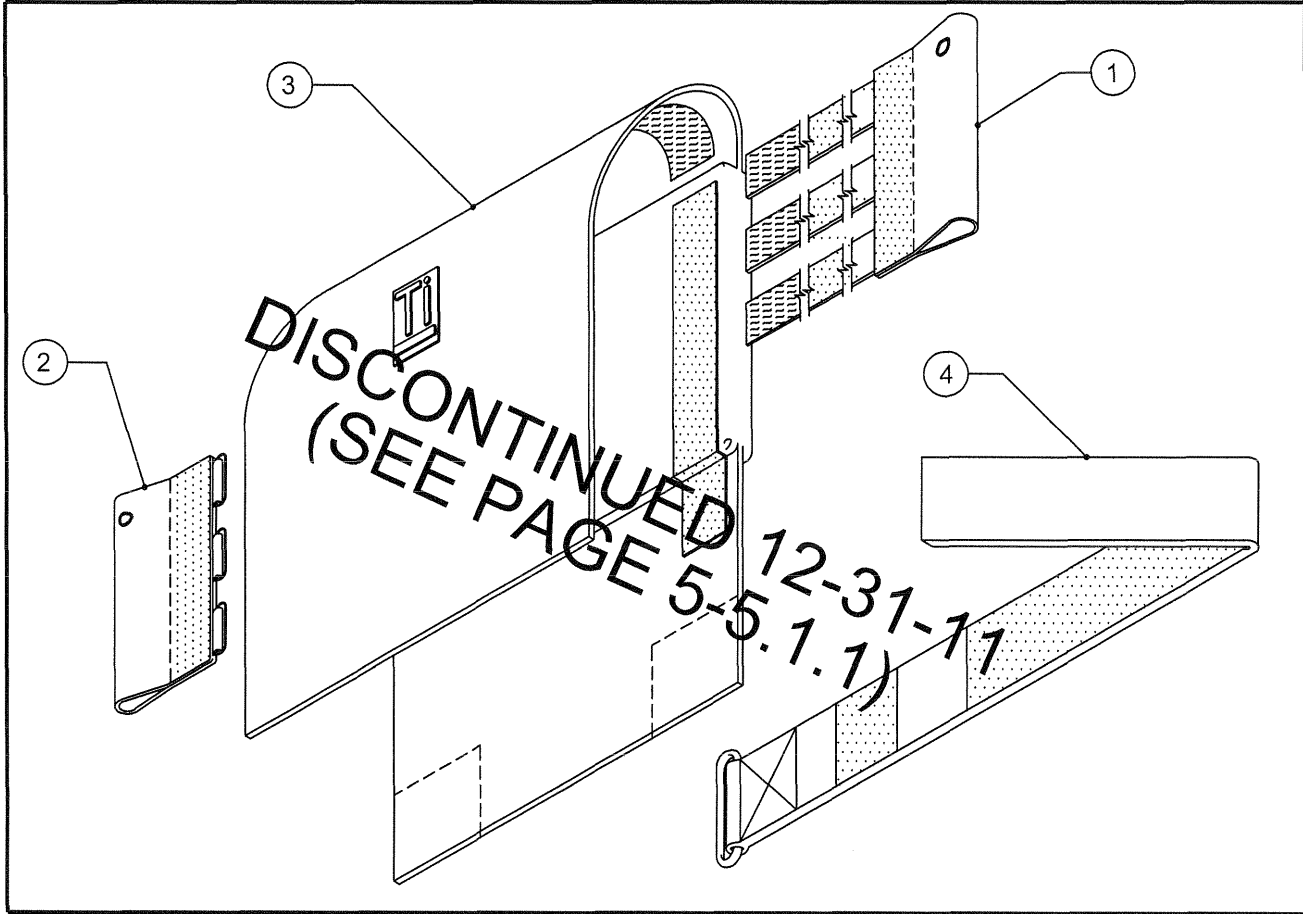

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE BELOW)

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-12.5"	STANDARD ADJ. HT 12.5"-16.5"	STANDARD ADJ. HT 16.5"-20.5"	ADJ. HT 9.5"-12.5" W/PUSH HANDLES	ADJ. HT 13.5"-16.5" W/PUSH HANDLES	ADJ. HT 17.5"-20.5" W/PUSH HANDLES
10"	AS103813	AS103815	AS103817	AS103819	AS103821	AS103823
11"	AS103814	AS103816	AS103818	AS103820	AS103822	AS103824
12"	AS101007	AS100132	AS100146	AS101014	AS100167	AS100181
13"	AS101008	AS100133	AS100147	AS101015	AS100168	AS100182
14"	AS101009	AS100134	AS100148	AS101016	AS100169	AS100183
15"	AS101010	AS100135	AS100149	AS101017	AS100170	AS100184
16"	AS101011	AS100136	AS100150	AS101018	AS100171	AS100185
17"	AS101012	AS100137	AS100151	AS101019	AS100172	AS100186
18"	AS101013	AS100138	AS100152	AS101020	AS100173	AS100187
19"	AS101554	AS101556	AS101560	AS101562	AS101564	AS101566
20"	AS101555	AS101557	AS101561	AS101563	AS101565	AS101567

BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ ZR, ZRA, Z10, AERO Z
TR(SERIES 2), YR(SERIES 2), AERO R, ZR(SERIES 2), ZRA(SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8.5"-12.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12.5"-16.5" BACK HEIGHTS USE TWO 2" BACK STRAPS

16.5"-20.5" BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-12.5"	STANDARD ADJ. HT 12.5"-16.5"	STANDARD ADJ. HT 16.5"-20.5"	ADJ. HT 9.5"-12.5" W/PUSH HANDLES	ADJ. HT 13.5"-16.5" W/PUSH HANDLES	ADJ. HT 17.5"-20.5" W/PUSH HANDLES
10"-11" RIGHT	CP102233	CP102234	CP102234	CP102235	CP102236	CP102236
10"-11" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
12"-13" RIGHT	CP101502	CP100049	CP100049	CP101496	CP100323	CP100323
12"-13" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
14"-15" RIGHT	CP101504	CP100264	CP100264	CP101497	CP100324	CP100324
14"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
16"-17" RIGHT	CP101505	CP100265	CP100265	CP101498	CP100325	CP100325
16"-17" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
18"-19" RIGHT	CP101506	CP100266	CP100266	CP101499	CP100326	CP100326
18"-19" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
20" RIGHT	CP101507	CP101508	CP101508	CP101500	CP101494	CP101494
20" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322

TABLE FOUR - BACK PAD TENSION ADJUSTABLE


CHAIR WIDTH	8.5"-12.5" SEAT BACK HEIGHT	12.5"-16.5" SEAT BACK HEIGHT	16.5"-20.5" SEAT BACK HEIGHT
10"	CP103574	CP103576	CP103578
11"	CP103575	CP103577	CP103579
12"	CP101253	CP100098	CP100883
13"	CP101254	CP100099	CP100884
14"	CP101255	CP100100	CP100885
15"	CP101256	CP100101	CP100886
16"	CP101257	CP100102	CP100887
17"	CP101258	CP100103	CP100888
18"	CP101259	CP100104	CP100889
19"	CP101532	CP101536	CP101488
20"	CP101533	CP101537	CP101489

DISCONTINUED 12-31-11
(SEE PAGE 5-5.1.2)



BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), ZR, ZRA, AERO Z, Z10, TR, YR, BB, AERO R, TR (SERIES 3), AERO T, TRA (SERIES 2)

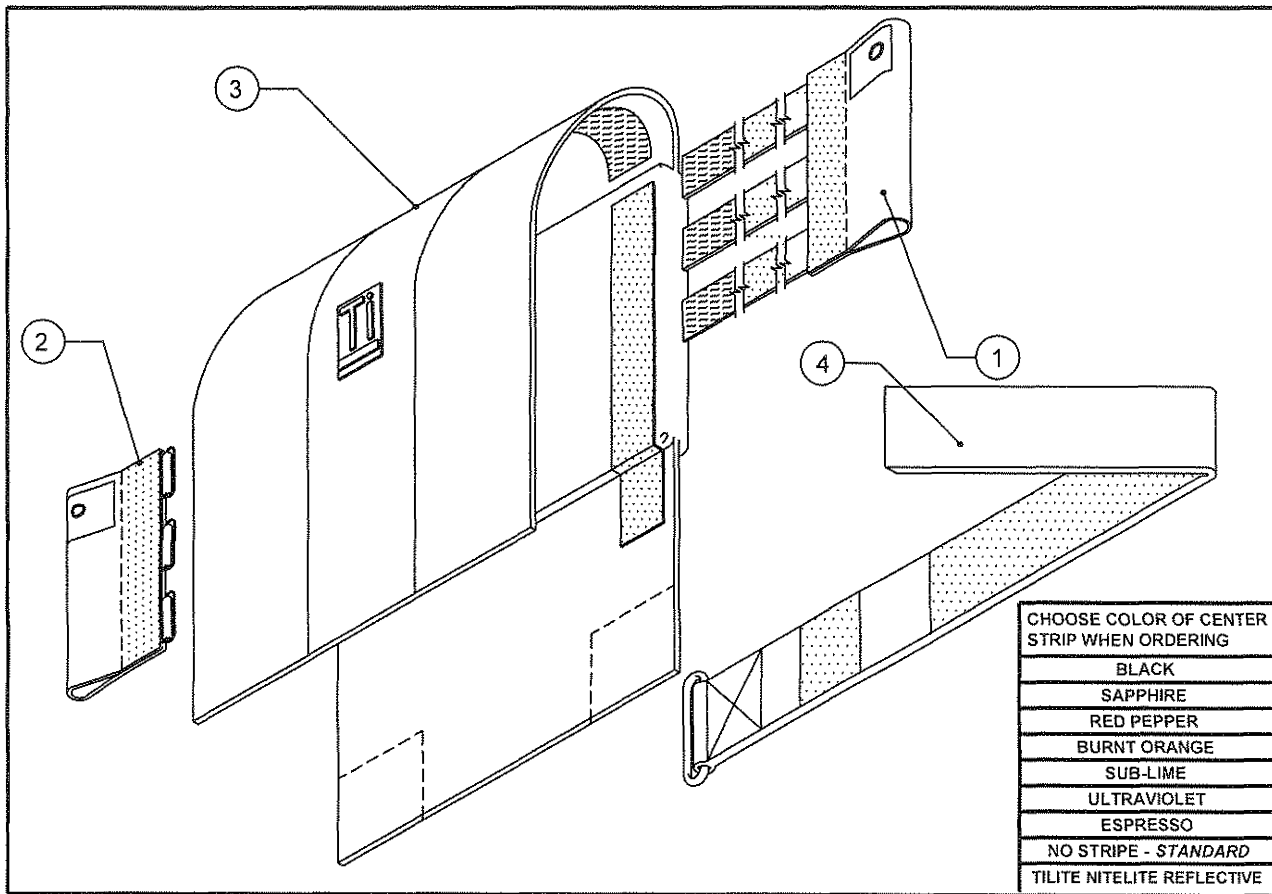

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE BELOW)

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"	AS103813	AS103815	AS103817	AS103819	AS103821	AS103823
11"	AS103814	AS103816	AS103818	AS103820	AS103822	AS103824
12"	AS101007	AS100132	AS100146	AS101014	AS100167	AS100181
13"	AS101008	AS100133	AS100147	AS101015	AS100168	AS100182
14"	AS101009	AS100134	AS100148	AS101016	AS100169	AS100183
15"	AS101010	AS100135	AS100149	AS101017	AS100170	AS100184
16"	AS101011	AS100136	AS100150	AS101018	AS100171	AS100185
17"	AS101012	AS100137	AS100151	AS101019	AS100172	AS100186
18"	AS101013	AS100138	AS100152	AS101020	AS100173	AS100187
19"	AS101554	AS101556	AS101560	AS101562	AS101564	AS101566
20"	AS101555	AS101557	AS101561	AS101563	AS101565	AS101567

BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ ZR (SERIES 2), ZRA (SERIES 2),
AERO Z (SERIES 2), ZR, ZRA, AERO Z, Z10, TR, YR, BB, AERO R, TR (SERIES 3),
AERO T, TRA (SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8.5"-11.5" (10"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (12"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" (16"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS


CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"-11" RIGHT	CP102233	CP102234	CP102234	CP102235	CP102236	CP102236
10"-11" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
12"-13" RIGHT	CP101502	CP100049	CP100049	CP101496	CP100323	CP100323
12"-13" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
14"-15" RIGHT	CP101504	CP100264	CP100264	CP101497	CP100324	CP100324
14"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
16"-17" RIGHT	CP101505	CP100265	CP100265	CP101498	CP100325	CP100325
16"-17" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
18"-19" RIGHT	CP101506	CP100266	CP100266	CP101499	CP100326	CP100326
18"-19" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
20" RIGHT	CP101507	CP101508	CP101508	CP101500	CP101494	CP101494
20" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	8.5"-11.5" BACK HEIGHT	12"-16.5" BACK HEIGHT	16"-20.5" BACK HEIGHT
	10"-12.5" BACK HEIGHT W/ PUSH	12"-16.5" BACK HEIGHT W/ PUSH	16"-20.5" BACK HEIGHT W/ PUSH
10"	CP103574	CP103576	CP103578
11"	CP103575	CP103577	CP103579
12"	CP101253	CP100098	CP100883
13"	CP101254	CP100099	CP100884
14"	CP101255	CP100100	CP100885
15"	CP101256	CP100101	CP100886
16"	CP101257	CP100102	CP100887
17"	CP101258	CP100103	CP100888
18"	CP101259	CP100104	CP100889
19"	CP101532	CP101536	CP101488
20"	CP101533	CP101537	CP101489

BREATHABLE BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), ZR, ZRA, AERO Z, Z10, TR, YR, BB, AERO R, TR (SERIES 3), AERO T, TRA (SERIES 2)

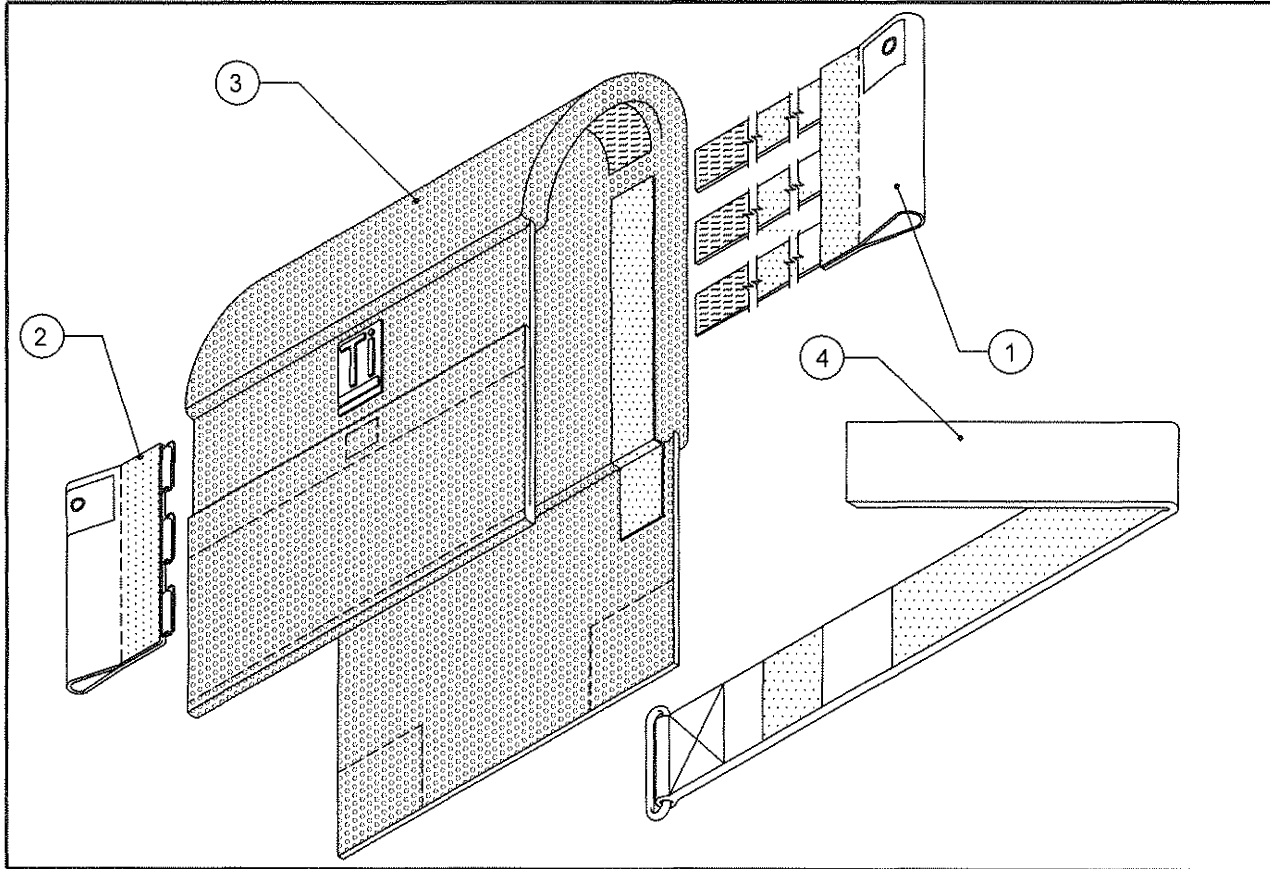

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOL. ASSEMBLY W/ PUSH (SEE BELOW)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE BELOW)

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"	AS115332	AS115343	AS115354	AS115365	AS115376	AS115387
11"	AS115333	AS115344	AS115355	AS115366	AS115377	AS115388
12"	AS115334	AS115345	AS115356	AS115367	AS115378	AS115389
13"	AS115335	AS115346	AS115357	AS115368	AS115379	AS115390
14"	AS115336	AS115347	AS115358	AS115369	AS115380	AS115391
15"	AS115337	AS115348	AS115359	AS115370	AS115381	AS115392
16"	AS115338	AS115349	AS115360	AS115371	AS115382	AS115393
17"	AS115339	AS115350	AS115361	AS115372	AS115383	AS115394
18"	AS115340	AS115351	AS115362	AS115373	AS115384	AS115395
19"	AS115341	AS115352	AS115363	AS115374	AS115385	AS115396
20"	AS115342	AS115353	AS115364	AS115375	AS115386	AS115397



BREATHABLE BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ ZR (SERIES 2), ZRA (SERIES 2),
AERO Z (SERIES 2), ZR, ZRA, AERO Z, Z10, TR, YR, BB, AERO R, TR (SERIES 3),
AERO T, TRA (SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8.5"-11.5" (10"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (12"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" (16"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS


CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"-11" RIGHT	CP102233	CP102234	CP102234	CP102235	CP102236	CP102236
10"-11" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
12"-13" RIGHT	CP101502	CP100049	CP100049	CP101496	CP100323	CP100323
12"-13" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
14"-15" RIGHT	CP101504	CP100264	CP100264	CP101497	CP100324	CP100324
14"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
16"-17" RIGHT	CP101505	CP100265	CP100265	CP101498	CP100325	CP100325
16"-17" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
18"-19" RIGHT	CP101506	CP100266	CP100266	CP101499	CP100326	CP100326
18"-19" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
20" RIGHT	CP101507	CP101508	CP101508	CP101500	CP101494	CP101494
20" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	8.5"-11.5" BACK HEIGHT	12"-16.5" BACK HEIGHT	16"-20.5" BACK HEIGHT
	10"-12.5" BACK HEIGHT W/ PUSH	12"-16.5" BACK HEIGHT W/ PUSH	16"-20.5" BACK HEIGHT W/ PUSH
10"	CP106976	CP106987	CP106998
11"	CP106977	CP106988	CP106999
12"	CP106978	CP106989	CP107000
13"	CP106979	CP106990	CP107001
14"	CP106980	CP106991	CP107002
15"	CP106981	CP106992	CP107003
16"	CP106982	CP106993	CP107004
17"	CP106983	CP106994	CP107005
18"	CP106984	CP106995	CP107006
19"	CP106985	CP106996	CP107007
20"	CP106986	CP106997	CP107008

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ ZR, ZRA, TR(SERIES 2), YR(SERIES 2),
AERO Z, AERO R, ZR(SERIES 2), ZRA(SERIES 2)

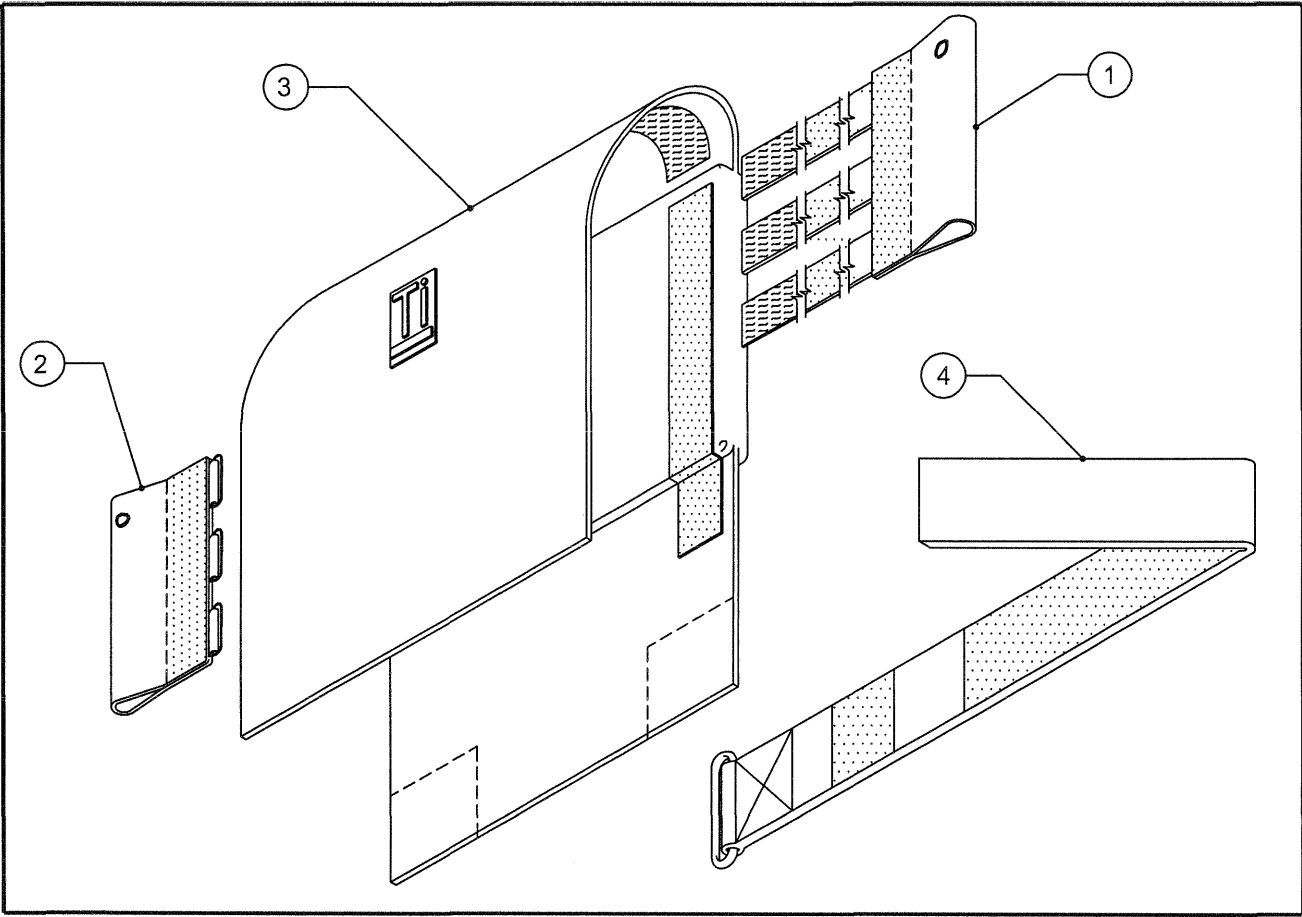

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 12.5"-14.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
10"	AS104001	AS104012	AS104023
11"	AS104002	AS104013	AS104024
12"	AS104003	AS104014	AS104025
13"	AS104004	AS104015	AS104026
14"	AS104005	AS104016	AS104027
15"	AS104006	AS104017	AS104028
16"	AS104007	AS104018	AS104029
17"	AS104008	AS104019	AS104030
18"	AS104009	AS104020	AS104031
19"	AS104010	AS104021	AS104032
20"	AS104011	AS104022	AS104033

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ ZR, TiLite™ ZRA, TR(SERIES 2), YR(SERIES 2), AERO Z, AERO R

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

12.5"-14.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

14.5"-19" BACK HEIGHTS USE TWO 2" BACK STRAPS

18.5"-21" BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 12.5"-14.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
10"-11" RIGHT	CP103685	CP103716	CP103716
10"-11" LEFT	CP103684	CP103715	CP103715
12"-13" RIGHT	CP103686	CP103717	CP103717
12"-13" LEFT	CP103684	CP103715	CP103715
14"-15" RIGHT	CP103687	CP103718	CP103718
14"-15" LEFT	CP103684	CP103715	CP103715
16"-17" RIGHT	CP103688	CP103719	CP103719
16"-17" LEFT	CP103684	CP103715	CP103715
18"-19" RIGHT	CP103689	CP103720	CP103720
18"-19" LEFT	CP103684	CP103715	CP103715
20" RIGHT	CP103690	CP103721	CP103721
20" LEFT	CP103684	CP103715	CP103715

TABLE FOUR - BACK PAD TENSION ADJUSTABLE


CHAIR WIDTH	12.5"-14.5" SEAT BACK HEIGHT	14.5"-19" SEAT BACK HEIGHT	18.5"-21" SEAT BACK HEIGHT
10"	CP103574	CP103576	CP103578
11"	CP103575	CP103577	CP103579
12"	CP101253	CP100098	CP100883
13"	CP101254	CP100099	CP100884
14"	CP101255	CP100100	CP100885
15"	CP101256	CP100101	CP100886
16"	CP101257	CP100102	CP100887
17"	CP101258	CP100103	CP100888
18"	CP101259	CP100104	CP100889
19"	CP101532	CP101536	CP101488
20"	CP101533	CP101537	CP101489

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES

TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)

ZR, ZRA, AERO Z, Z10, TR, YR, TR (SERIES 3), AERO T, TRA (SERIES 2)

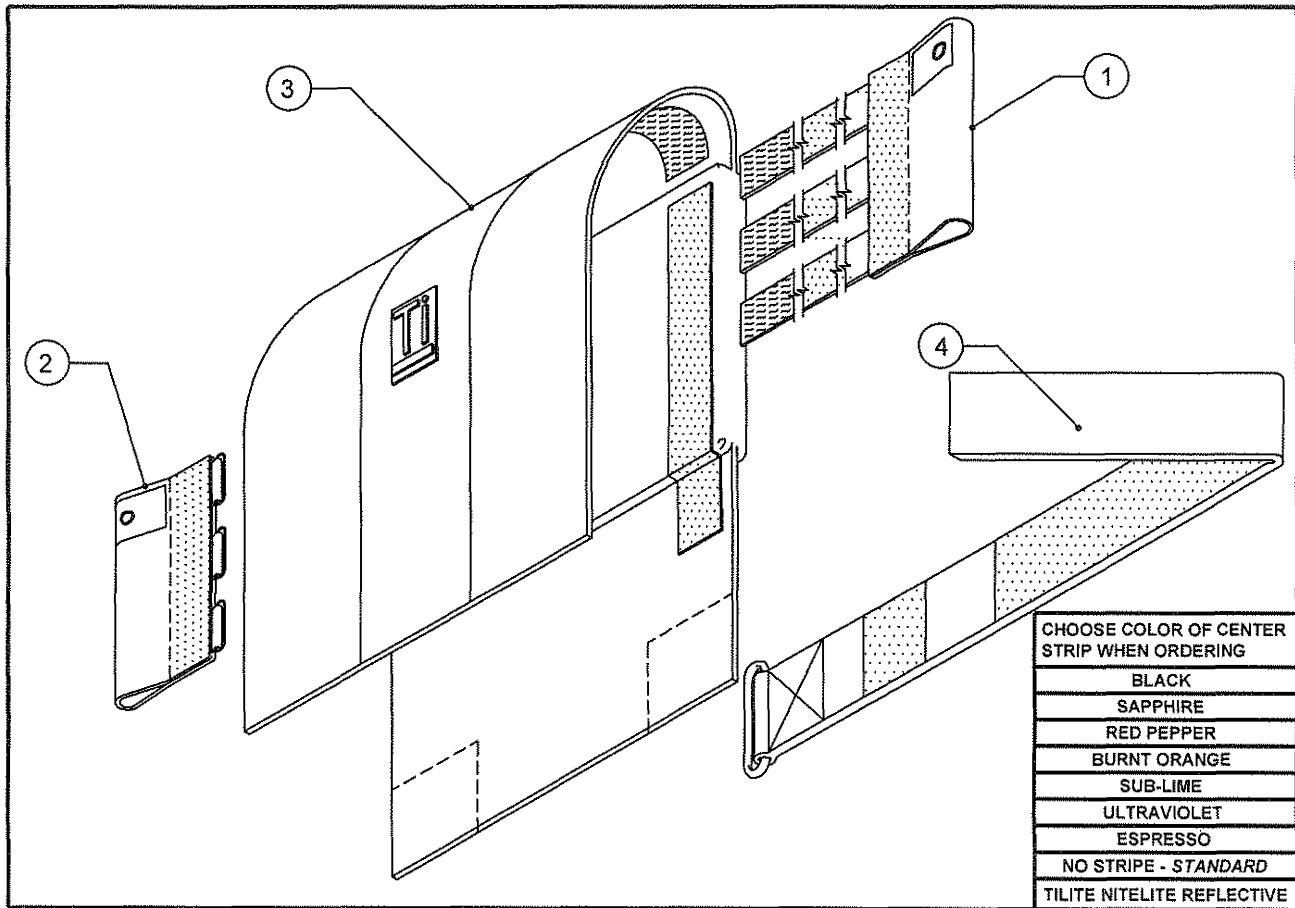

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 13"-15.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
10"	AS104001	AS104012	AS104023
11"	AS104002	AS104013	AS104024
12"	AS104003	AS104014	AS104025
13"	AS104004	AS104015	AS104026
14"	AS104005	AS104016	AS104027
15"	AS104006	AS104017	AS104028
16"	AS104007	AS104018	AS104029
17"	AS104008	AS104019	AS104030
18"	AS104009	AS104020	AS104031
19"	AS104010	AS104021	AS104032
20"	AS104011	AS104022	AS104033

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES

TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2)
ZR, ZRA, AERO Z, Z10, TR, YR, TR (SERIES 3), AERO T, TRA (SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

13"-15.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

14.5"-19" BACK HEIGHTS USE TWO 2" BACK STRAPS

18.5"-21" BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 13"-15.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
10"-11" RIGHT	CP103685	CP103716	CP103716
10"-11" LEFT	CP103684	CP103715	CP103715
12"-13" RIGHT	CP103686	CP103717	CP103717
12"-13" LEFT	CP103684	CP103715	CP103715
14"-15" RIGHT	CP103687	CP103718	CP103718
14"-15" LEFT	CP103684	CP103715	CP103715
16"-17" RIGHT	CP103688	CP103719	CP103719
16"-17" LEFT	CP103684	CP103715	CP103715
18"-19" RIGHT	CP103689	CP103720	CP103720
18"-19" LEFT	CP103684	CP103715	CP103715
20" RIGHT	CP103690	CP103721	CP103721
20" LEFT	CP103684	CP103715	CP103715


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	13"-15.5" SEAT BACK HEIGHT	14.5"-19" SEAT BACK HEIGHT	18.5"-21" SEAT BACK HEIGHT
10"	CP103574	CP103576	CP103578
11"	CP103575	CP103577	CP103579
12"	CP101253	CP100098	CP100883
13"	CP101254	CP100099	CP100884
14"	CP101255	CP100100	CP100885
15"	CP101256	CP100101	CP100886
16"	CP101257	CP100102	CP100887
17"	CP101258	CP100103	CP100888
18"	CP101259	CP100104	CP100889
19"	CP101532	CP101536	CP101488
20"	CP101533	CP101537	CP101489

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES

TiLite™ ZR, ZRA, TR (SERIES 3), YR (SERIES 2), AERO Z (SERIES 2),
AERO R, ZR (SERIES 2), ZRA (SERIES 2), AERO T, TRA (SERIES 2)

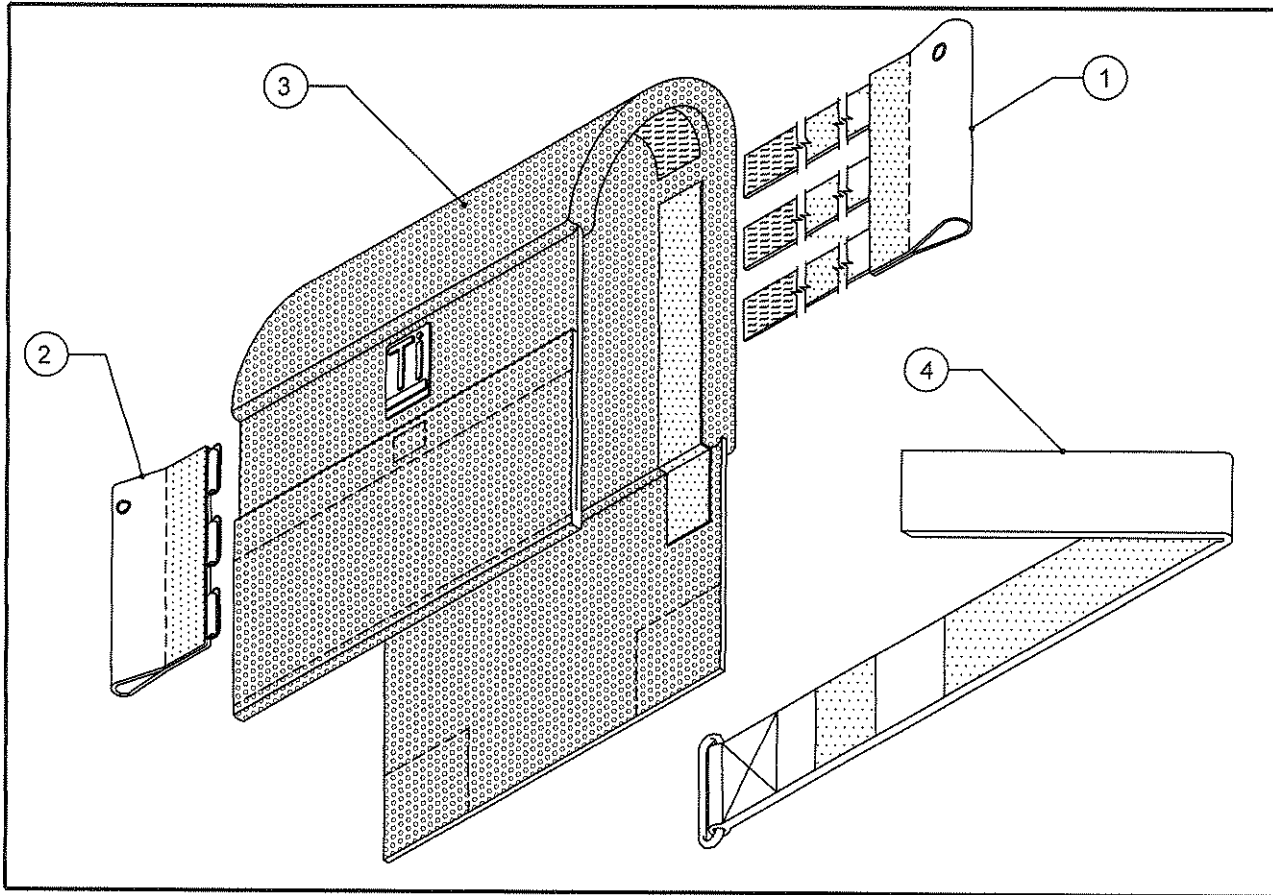

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 12.5"-14.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
10"	AS115299	AS115310	AS115321
11"	AS115300	AS115311	AS115322
12"	AS115301	AS115312	AS115323
13"	AS115302	AS115313	AS115324
14"	AS115303	AS115314	AS115325
15"	AS115304	AS115315	AS115326
16"	AS115305	AS115316	AS115327
17"	AS115306	AS115317	AS115328
18"	AS115307	AS115318	AS115329
19"	AS115308	AS115319	AS115330
20"	AS115309	AS115320	AS115331

BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ ZR, TiLite™ ZRA, TR (SERIES 2), YR (SERIES 2),
AERO Z, AERO R, ZR (SERIES 2), ZRA (SERIES 2), AERO T, TRA (SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

12.5"-14.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

14.5"-19" BACK HEIGHTS USE TWO 2" BACK STRAPS

18.5"-21" BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS


CHAIR WIDTH	ADJ. HEIGHT 12.5"-14.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
10"-11" RIGHT	CP103685	CP103716	CP103716
10"-11" LEFT	CP103684	CP103715	CP103715
12"-13" RIGHT	CP103686	CP103717	CP103717
12"-13" LEFT	CP103684	CP103715	CP103715
14"-15" RIGHT	CP103687	CP103718	CP103718
14"-15" LEFT	CP103684	CP103715	CP103715
16"-17" RIGHT	CP103688	CP103719	CP103719
16"-17" LEFT	CP103684	CP103715	CP103715
18"-19" RIGHT	CP103689	CP103720	CP103720
18"-19" LEFT	CP103684	CP103715	CP103715
20" RIGHT	CP103690	CP103721	CP103721
20" LEFT	CP103684	CP103715	CP103715

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	12.5"-14.5" SEAT BACK HEIGHT	14.5"-19" SEAT BACK HEIGHT	18.5"-21" SEAT BACK HEIGHT
10"	CP106976	CP106987	CP106998
11"	CP106977	CP106988	CP106999
12"	CP106978	CP106989	CP107000
13"	CP106979	CP106990	CP107001
14"	CP106980	CP106991	CP107002
15"	CP106981	CP106992	CP107003
16"	CP106982	CP106993	CP107004
17"	CP106983	CP106994	CP107005
18"	CP106984	CP106995	CP107006
19"	CP106985	CP106996	CP107007
20"	CP106986	CP106997	CP107008

BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ TWIST

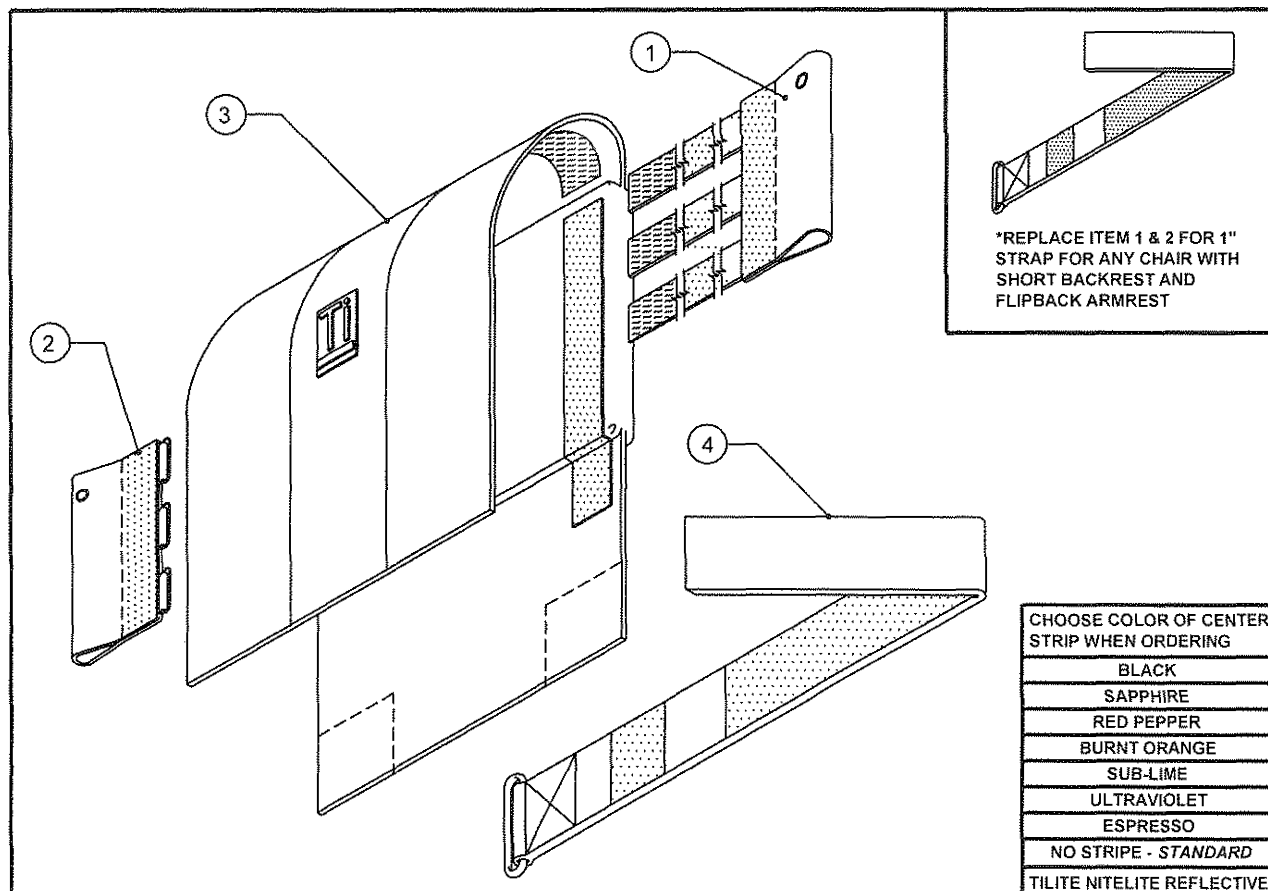

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE BELOW)

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10.5"-12.5" W/PUSH HANDLES	ADJ. HT 13.5"-16.5" W/PUSH HANDLES	ADJ. HT 17.5"-20.5" W/PUSH HANDLES
8"-10"	AS111813	AS111819		AS111827	AS111833	
9"-11"	AS111814	AS111820		AS111828	AS111834	
10"-12"	AS111815	AS111821		AS111829	AS111835	
11"-13"	AS111816	AS111822		AS111830	AS111836	
12"-14"	AS111817	AS111823	AS111825	AS111831	AS111837	AS111839
13"-15"	AS111818	AS111824	AS111826	AS111832	AS111838	AS111840

BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ TWIST

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	8"-10" CHAIR WIDTH	9"-11" CHAIR WIDTH	10"-12" CHAIR WIDTH	11"-13" CHAIR WIDTH	12"-14" CHAIR WIDTH	13"-15" CHAIR WIDTH
1" WIDE	CP105515	CP105516	CP105517	CP105518	CP105519	CP105520
2" WIDE	CP105521	CP105522	CP105523	CP105524	CP105525	CP105526

*NOTE: FOR ITEM 4

8.5"-11.5" (10.5"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (13.5"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" (17.5"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS


CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10.5"-12.5" W/PUSH HANDLES	ADJ. HT 13.5"-16.5" W/PUSH HANDLES	ADJ. HT 17.5"-20.5" W/PUSH HANDLES
8"-10" RIGHT	CP105527	CP105533		CP105539	CP105545	
8"-10" LEFT	CP101503	CP100050		CP101495	CP100322	
9"-11" RIGHT	CP105528	CP105534		CP105540	CP105546	
9"-11" LEFT	CP101503	CP100050		CP101495	CP100322	
10"-12" RIGHT	CP105529	CP105535		CP105541	CP105547	
10"-12" LEFT	CP101503	CP100050		CP101495	CP100322	
11"-13" RIGHT	CP105530	CP105536		CP105542	CP105548	
11"-13" LEFT	CP101503	CP100050		CP101495	CP100322	
12"-14" RIGHT	CP105531	CP105537	CP105537	CP105543	CP105549	CP105549
12"-14" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
13"-15" RIGHT	CP105532	CP105538	CP105538	CP105544	CP105550	CP105550
13"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	8.5"-11.5" BACK HEIGHT	12"-16.5" BACK HEIGHT	16"-20.5" BACK HEIGHT
	10.5"-12.5" BACK HEIGHT W/ PUSH	13.5"-16.5" BACK HEIGHT W/ PUSH	17.5"-20.5" BACK HEIGHT W/ PUSH
8"-10"	CP105551	CP105557	
9"-11"	CP105552	CP105558	
10"-12"	CP105553	CP105559	
11"-13"	CP105554	CP105560	
12"-14"	CP105555	CP105561	CP105563
13"-15"	CP105556	CP105562	CP105564

BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ TWIST WITH FOLDING PUSH HANDLES

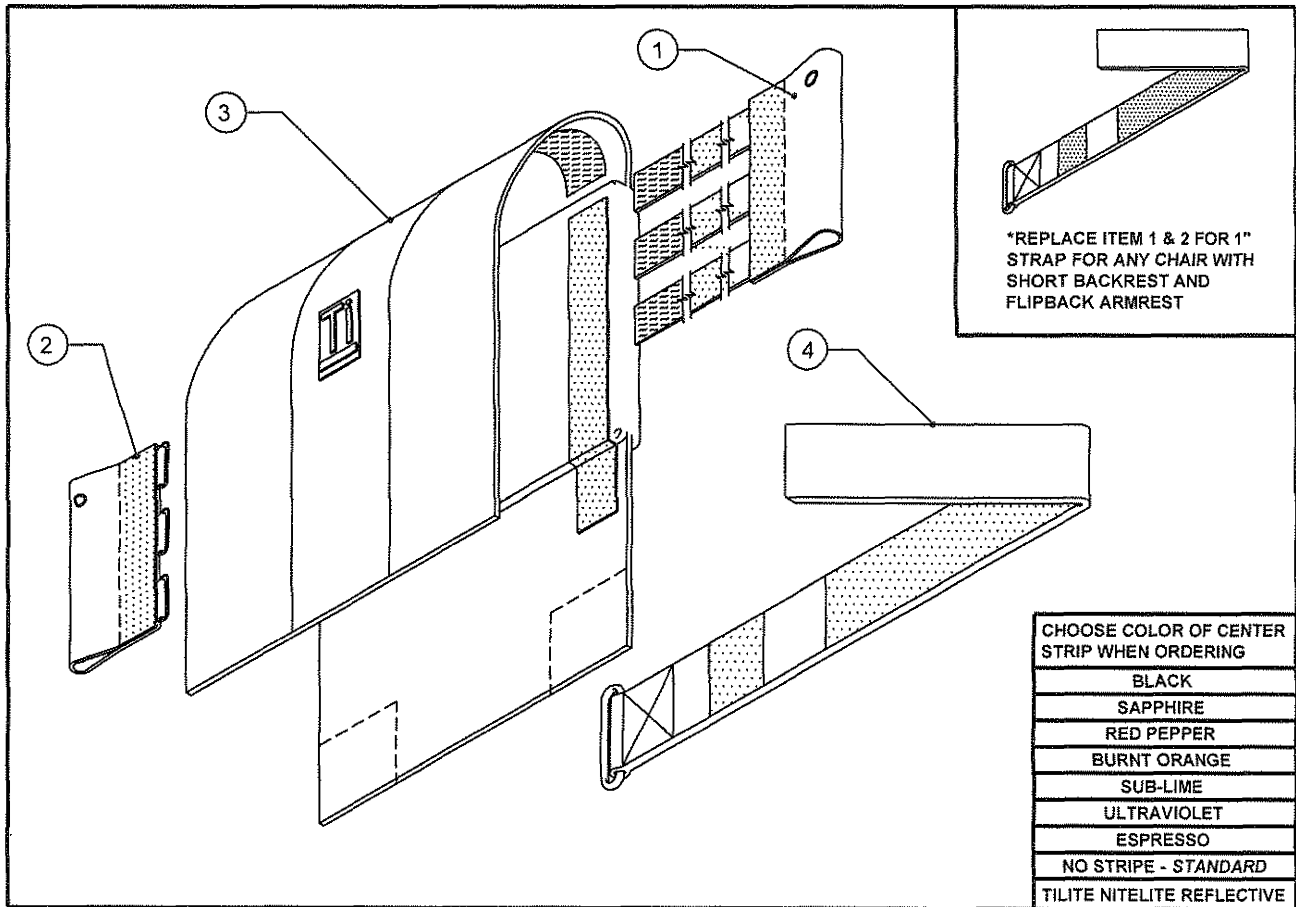

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE BELOW)

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	13.5"-15.5" SEAT BACK HEIGHT	14.5"-19" SEAT BACK HEIGHT	18.5"-21"- SEAT BACK HEIGHT
8"-10"	AS111841	AS111847	
9"-11"	AS111842	AS111848	
10"-12"	AS111843	AS111849	
11"-13"	AS111844	AS111850	
12"-14"	AS111845	AS111851	AS111853
13"-15"	AS111846	AS111852	AS111854

BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ TWIST

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	8"-10" CHAIR WIDTH	9"-11" CHAIR WIDTH	10"-12" CHAIR WIDTH	11"-13" CHAIR WIDTH	12"-14" CHAIR WIDTH	13"-15" CHAIR WIDTH
1" WIDE	CP105515	CP105516	CP105517	CP105518	CP105519	CP105520
2" WIDE	CP105521	CP105522	CP105523	CP105524	CP105525	CP105526

*NOTE: FOR ITEM 4

8.5"-11.5" (10.5"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (13.5"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" (17.5"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 13"-15.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
8"-10" RIGHT	CP105565	CP105571	
8"-10" LEFT	CP103684	CP103715	
9"-11" RIGHT	CP105566	CP105572	
9"-11" LEFT	CP103684	CP103715	
10"-12" RIGHT	CP105567	CP105573	
10"-12" LEFT	CP103684	CP103715	
11"-13" RIGHT	CP105568	CP105574	
11"-13" LEFT	CP103684	CP103715	
12"-14" RIGHT	CP105569	CP105575	CP105575
12"-14" LEFT	CP103684	CP103715	CP103715
13"-15" RIGHT	CP105570	CP105576	CP105576
13"-15" LEFT	CP103684	CP103715	CP103715


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	13.5"-15.5" SEAT BACK HEIGHT	14.5"-19" SEAT BACK HEIGHT	18.5"-21" SEAT BACK HEIGHT
8"-10"	CP105551	CP105557	
9"-11"	CP105552	CP105558	
10"-12"	CP105553	CP105559	
11"-13"	CP105554	CP105560	
12"-14"	CP105555	CP105561	CP105563
13"-15"	CP105556	CP105562	CP105564



BREATHABLE BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ TWIST

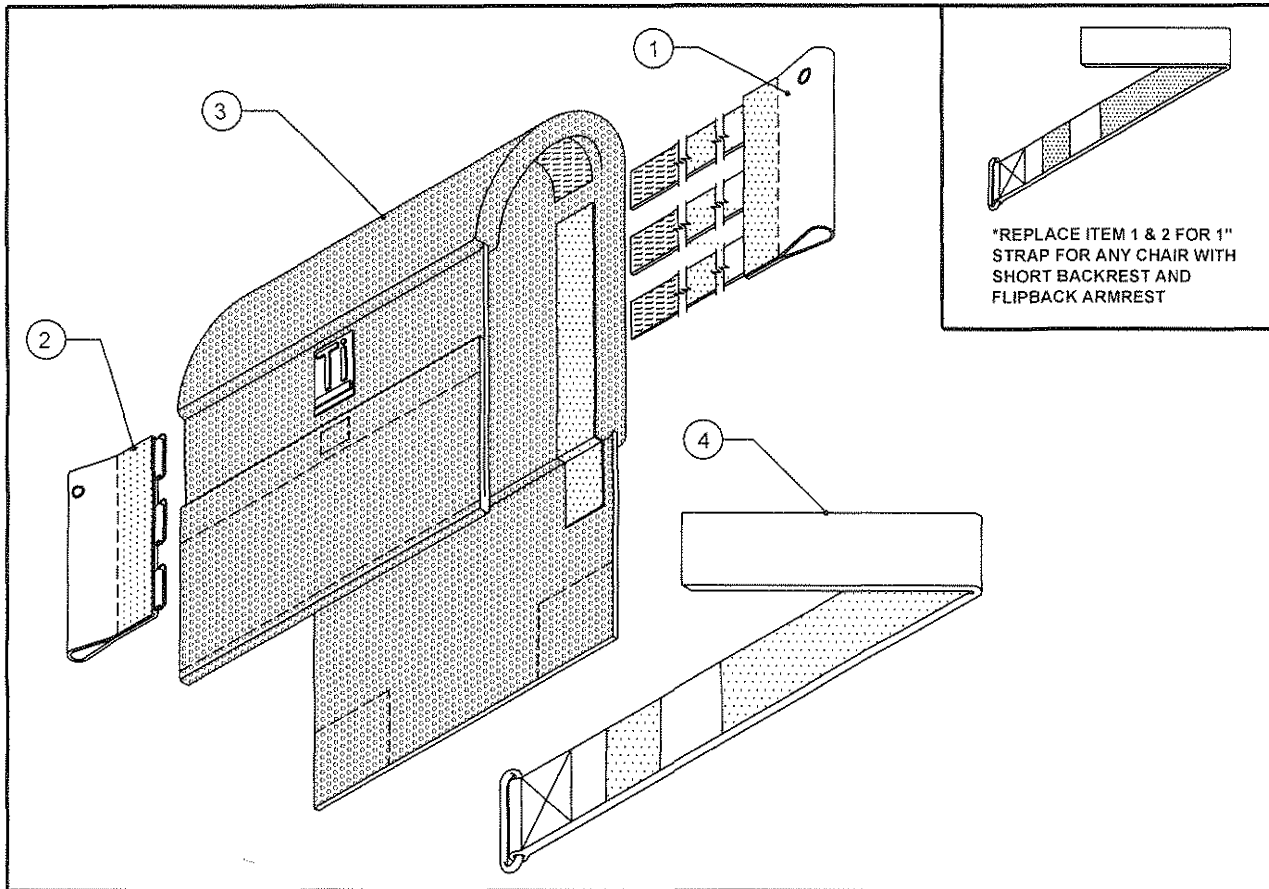

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOL. ASSEMBLY W/ PUSH (SEE BELOW)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE BELOW)

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10.5"-12.5" W/PUSH HANDLES	ADJ. HT 13.5"-16.5" W/PUSH HANDLES	ADJ. HT 17.5"-20.5" W/PUSH HANDLES
8"-10"	AS115398	AS115404		AS115412	AS115418	
9"-11"	AS115399	AS115405		AS115413	AS115419	
10"-12"	AS115400	AS115406		AS115414	AS115420	
11"-13"	AS115401	AS115407		AS115415	AS115421	
12"-14"	AS115402	AS115408	AS115410	AS115416	AS115422	AS115424
13"-15"	AS115403	AS115409	AS115411	AS115417	AS115423	AS115425



BREATHABLE BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ TWIST

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	8"-10" CHAIR WIDTH	9"-11" CHAIR WIDTH	10"-12" CHAIR WIDTH	11"-13" CHAIR WIDTH	12"-14" CHAIR WIDTH	13"-15" CHAIR WIDTH
1" WIDE	CP105515	CP105516	CP105517	CP105518	CP105519	CP105520
2" WIDE	CP105521	CP105522	CP105523	CP105524	CP105525	CP105526

*NOTE: FOR ITEM 4

8.5"-11.5" (10.5"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (13.5"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" (17.5"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10.5"-12.5" W/PUSH HANDLES	ADJ. HT 13.5"-16.5" W/PUSH HANDLES	ADJ. HT 17.5"-20.5" W/PUSH HANDLES
8"-10" RIGHT	CP105527	CP105533		CP105539	CP105545	
8"-10" LEFT	CP101503	CP100050		CP101495	CP100322	
9"-11" RIGHT	CP105528	CP105534		CP105540	CP105546	
9"-11" LEFT	CP101503	CP100050		CP101495	CP100322	
10"-12" RIGHT	CP105529	CP105535		CP105541	CP105547	
10"-12" LEFT	CP101503	CP100050		CP101495	CP100322	
11"-13" RIGHT	CP105530	CP105536		CP105542	CP105548	
11"-13" LEFT	CP101503	CP100050		CP101495	CP100322	
12"-14" RIGHT	CP105531	CP105537	CP105537	CP105543	CP105549	CP105549
12"-14" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
13"-15" RIGHT	CP105532	CP105538	CP105538	CP105544	CP105550	CP105550
13"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	8.5"-11.5" BACK HEIGHT	12"-16.5" BACK HEIGHT	16"-20.5" BACK HEIGHT
	10.5"-12.5" BACK HEIGHT W/ PUSH	13.5"-16.5" BACK HEIGHT W/ PUSH	17.5"-20.5" BACK HEIGHT W/ PUSH
8"-10"	CP107009	CP107015	
9"-11"	CP107010	CP107016	
10"-12"	CP107011	CP107017	
11"-13"	CP107012	CP107018	
12"-14"	CP107013	CP107019	CP107021
13"-15"	CP107014	CP107020	CP107022



BREATHABLE BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ TWIST WITH FOLDING PUSH HANDLES

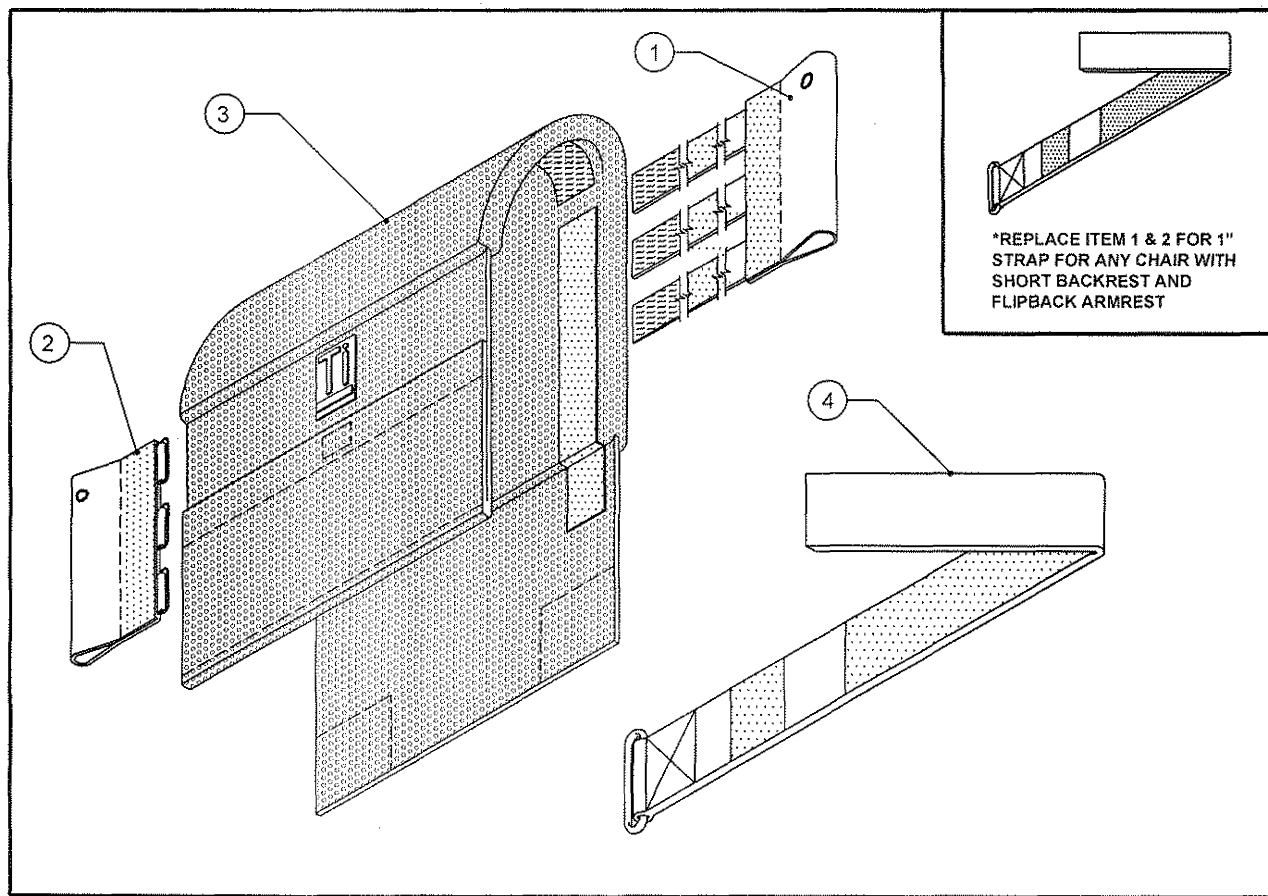

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE BELOW)

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	13.5"-15.5" SEAT BACK HEIGHT	14.5"-19" SEAT BACK HEIGHT	18.5"-21" SEAT BACK HEIGHT
8"-10"	AS115426	AS115432	
9"-11"	AS115427	AS115433	
10"-12"	AS115428	AS115434	
11"-13"	AS115429	AS115435	
12"-14"	AS115430	AS115436	AS115438
13"-15"	AS115431	AS115437	AS115439

BREATHABLE BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ TWIST

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	8"-10" CHAIR WIDTH	9"-11" CHAIR WIDTH	10"-12" CHAIR WIDTH	11"-13" CHAIR WIDTH	12"-14" CHAIR WIDTH	13"-15" CHAIR WIDTH
1" WIDE	CP105515	CP105516	CP105517	CP105518	CP105519	CP105520
2" WIDE	CP105521	CP105522	CP105523	CP105524	CP105525	CP105526

*NOTE: FOR ITEM 4

8.5"-11.5" (10.5"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (13.5"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" (17.5"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS

TABLE THREE - BACKREST EARS


CHAIR WIDTH	ADJ. HEIGHT 13"-15.5"	ADJ. HEIGHT 14.5"-19"	ADJ. HEIGHT 18.5"-21"
8"-10" RIGHT	CP105565	CP105571	
8"-10" LEFT	CP103684	CP103715	
9"-11" RIGHT	CP105566	CP105572	
9"-11" LEFT	CP103684	CP103715	
10"-12" RIGHT	CP105567	CP105573	
10"-12" LEFT	CP103684	CP103715	
11"-13" RIGHT	CP105568	CP105574	
11"-13" LEFT	CP103684	CP103715	
12"-14" RIGHT	CP105569	CP105575	CP105575
12"-14" LEFT	CP103684	CP103715	CP103715
13"-15" RIGHT	CP105570	CP105576	CP105576
13"-15" LEFT	CP103684	CP103715	CP103715

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	13.5"-15.5" SEAT BACK HEIGHT	14.5"-19" SEAT BACK HEIGHT	18.5"-21" SEAT BACK HEIGHT
8"-10"	CP107009	CP107015	
9"-11"	CP107010	CP107016	
10"-12"	CP107011	CP107017	
11"-13"	CP107012	CP107018	
12"-14"	CP107013	CP107019	CP107021
13"-15"	CP107014	CP107020	CP107022

BACK UPHOLSTERY

VELCRO ADJUSTABLE FOR TiLite™ ZR, ZRA, TR(SERIES 2), YR(SERIES 2), Aero Z, Aero R, Z10, ZR(SERIES 2), ZRA(SERIES 2)

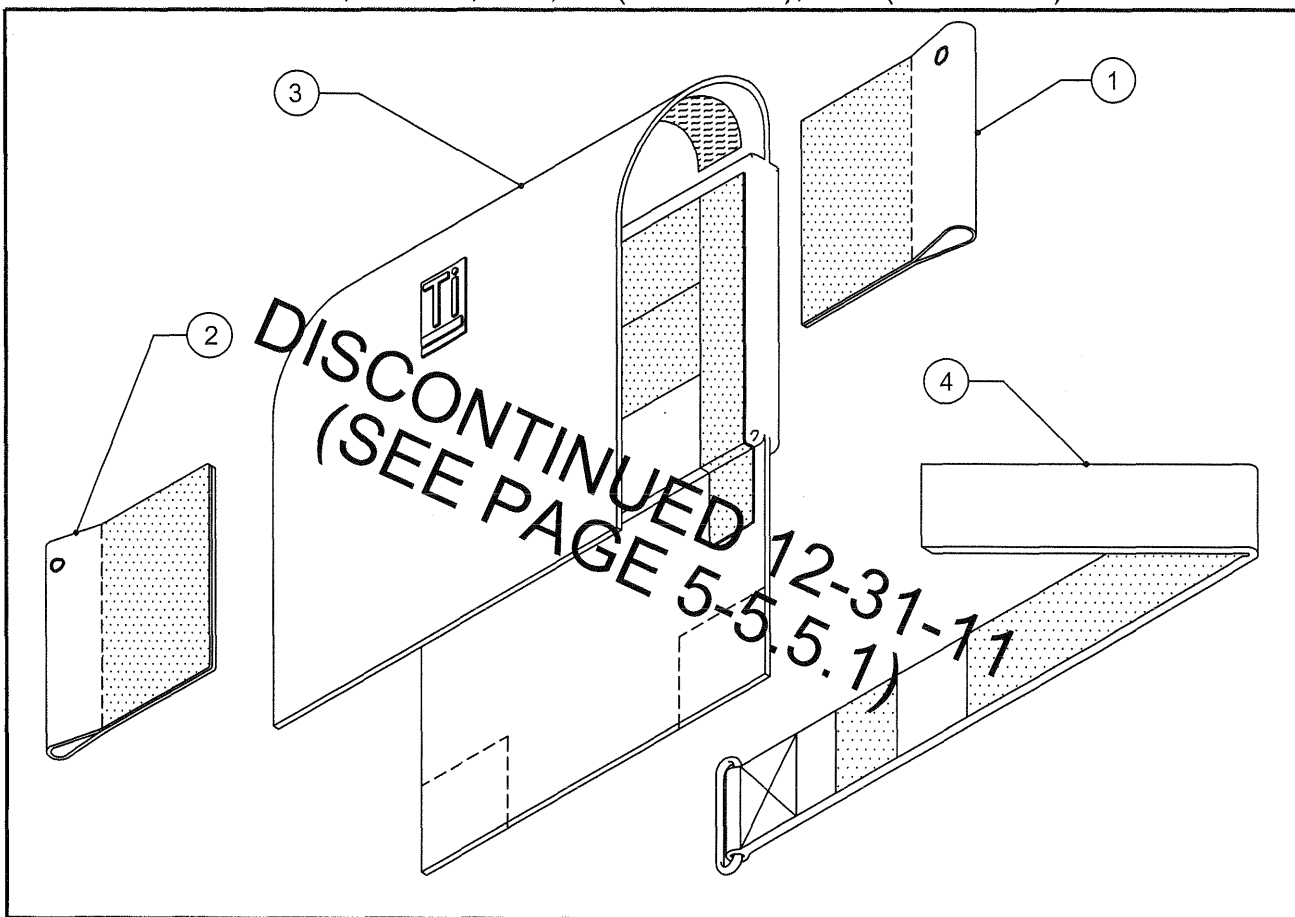

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4	1	TABLE ONE	VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-12.5"	STANDARD ADJ. HT 12.5"-16.5"	STANDARD ADJ. HT 16.5"-20.5"	ADJ. HT 9.5"-12.5" W/PUSH HANDLES	ADJ. HT 13.5"-16.5" W/PUSH HANDLES	ADJ. HT 17.5"-20.5" W/PUSH HANDLES
10"	AS103801	AS103803	AS103805	AS103807	AS103809	AS103811
11"	AS103802	AS103804	AS103806	AS103808	AS103810	AS103812
12"	AS100979	AS100125	AS100993	AS100986	AS100160	AS101000
13"	AS100980	AS100126	AS100994	AS100987	AS100161	AS101001
14"	AS100981	AS100127	AS100995	AS100988	AS100162	AS101002
15"	AS100982	AS100128	AS100996	AS100989	AS100163	AS101003
16"	AS100983	AS100129	AS100997	AS100990	AS100164	AS101004
17"	AS100984	AS100130	AS100998	AS100991	AS100165	AS101005
18"	AS100985	AS100131	AS100999	AS100992	AS100166	AS101006
19"	AS101533	AS101535	AS101546	AS101548	AS101550	AS101552
20"	AS101534	AS101536	AS101547	AS101549	AS101551	AS101553

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8"-12" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16" BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20" BACK HEIGHTS USE THREE 2" BACK STRAPS



BACK UPHOLSTERY (CONT.)

VELCRO ADJUSTABLE FOR TiLite™ ZR, ZRA, TR(SERIES 2), YR(SERIES 2),
Aero Z, Aero R, Z10, ZR(SERIES 2), ZRA(SERIES 2)

TABLE THREE - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
10"-11" RIGHT	CP102238	CP102241	CP102241	CP102239	CP102243	CP102243
10"-11" LEFT	CP102237	CP102242	CP102242	CP102240	CP102244	CP102244
12"-13" RIGHT	CP101510	CP100450	CP100450	CP101484	CP100508	CP100508
12"-13" LEFT	CP101509	CP100514	CP100514	CP101485	CP100673	CP100673
14"-18" RIGHT	CP101511	CP100451	CP100451	CP101486	CP100509	CP100509
14"-18" LEFT	CP101512	CP100454	CP100454	CP101487	CP100507	CP100507
19"-20" RIGHT	CP101524	CP101526	CP101526	CP101530	CP101528	CP101528
19"-20" LEFT	CP101523	CP101525	CP101525	CP101529	CP101527	CP101527


TABLE FOUR - BACK PAD VELCRO ADJUSTABLE

CHAIR WIDTH	8"-12" SEAT BACK HEIGHT	12"-16" SEAT BACK HEIGHT	16"-20" SEAT BACK HEIGHT
10"	CP103568	CP103570	CP103572
11"	CP103569	CP103571	CP103573
12"	CP101205	CP100460	CP100890
13"	CP101206	CP100461	CP100891
14"	CP101207	CP100462	CP100892
15"	CP101208	CP100463	CP100893
16"	CP101209	CP100464	CP100894
17"	CP101210	CP100465	CP100895
18"	CP101211	CP100466	CP100896
19"	CP101534	CP101538	CP101540
20"	CP101535	CP101539	CP101541

DISCONTINUED 12-31-11
(SEE PAGE 5-5.5.2)

BACK UPHOLSTERY

VELCRO ADJUSTABLE FOR TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), ZR, ZRA, AERO Z, Z10, TR, YR, BB, AERO R, TR (SERIES 3), AERO T, TRA (SERIES 2)

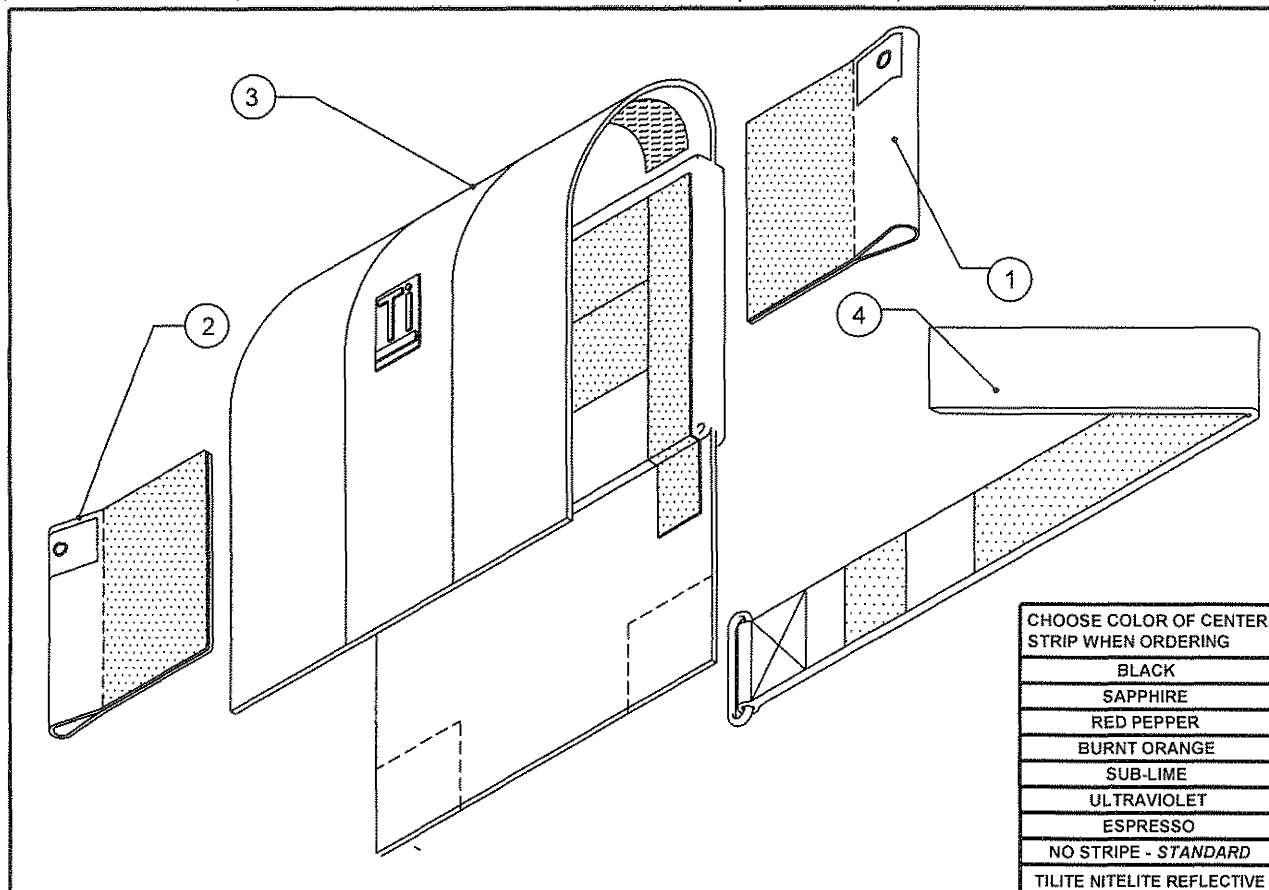

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4**		TABLE ONE	VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"	AS103801	AS103803	AS103805	AS103807	AS103809	AS103811
11"	AS103802	AS103804	AS103806	AS103808	AS103810	AS103812
12"	AS100979	AS100125	AS100993	AS100986	AS100160	AS101000
13"	AS100980	AS100126	AS100994	AS100987	AS100161	AS101001
14"	AS100981	AS100127	AS100995	AS100988	AS100162	AS101002
15"	AS100982	AS100128	AS100996	AS100989	AS100163	AS101003
16"	AS100983	AS100129	AS100997	AS100990	AS100164	AS101004
17"	AS100984	AS100130	AS100998	AS100991	AS100165	AS101005
18"	AS100985	AS100131	AS100999	AS100992	AS100166	AS101006
19"	AS101533	AS101535	AS101546	AS101548	AS101550	AS101552
20"	AS101534	AS101536	AS101547	AS101549	AS101551	AS101553

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8.5"-11.5" (10"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (12"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" (16"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS



BACK UPHOLSTERY (CONT.)

VELCRO ADJUSTABLE FOR TiLite™ ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), ZR, ZRA, AERO Z, Z10, TR, YR, BB, AERO R, TR (SERIES 3), AERO T, TRA (SERIES 2)

TABLE THREE - BACKREST EARS


CHAIR WIDTH	STANDARD ADJ. HEIGHT 8.5"-11.5"	STANDARD ADJ. HEIGHT 12"-16.5"	STANDARD ADJ. HEIGHT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"-11" RIGHT	CP102238	CP102241	CP102241	CP102239	CP102243	CP102243
10"-11" LEFT	CP102237	CP102242	CP102242	CP102240	CP102244	CP102244
12"-13" RIGHT	CP101510	CP100450	CP100450	CP101484	CP100508	CP100508
12"-13" LEFT	CP101509	CP100514	CP100514	CP101485	CP100673	CP100673
14"-18" RIGHT	CP101511	CP100451	CP100451	CP101486	CP100509	CP100509
14"-18" LEFT	CP101512	CP100454	CP100454	CP101487	CP100507	CP100507
19"-20" RIGHT	CP101524	CP101526	CP101526	CP101530	CP101528	CP101528
19"-20" LEFT	CP101523	CP101525	CP101525	CP101529	CP101527	CP101527

TABLE FOUR - BACK PAD VELCRO ADJUSTABLE

CHAIR WIDTH	8.5"-11.5" BACK HEIGHT	12"-16.5" BACK HEIGHT	16"-20.5" BACK HEIGHT
	10"-12.5" BACK HEIGHT W/ PUSH	12"-16.5" BACK HEIGHT W/ PUSH	16"-20.5" BACK HEIGHT W/ PUSH
10"	CP103568	CP103570	CP103572
11"	CP103569	CP103571	CP103573
12"	CP101205	CP100460	CP100890
13"	CP101206	CP100461	CP100891
14"	CP101207	CP100462	CP100892
15"	CP101208	CP100463	CP100893
16"	CP101209	CP100464	CP100894
17"	CP101210	CP100465	CP100895
18"	CP101211	CP100466	CP100896
19"	CP101534	CP101538	CP101540
20"	CP101535	CP101539	CP101541

BACK UPHOLSTERY

VELCRO ADJUSTABLE FOR TiLite™ TWIST

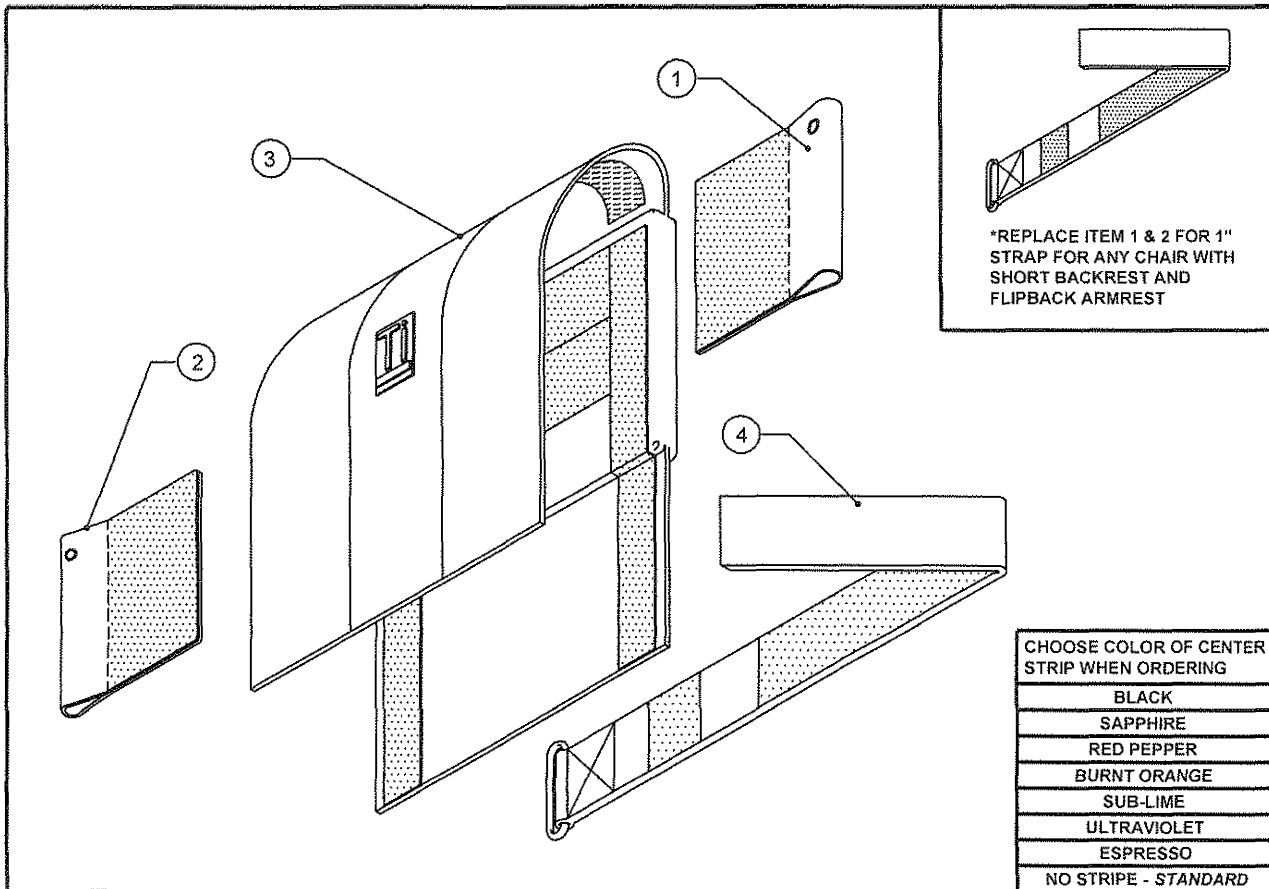

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4**		TABLE ONE	VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HT 8.5"-11.5"	STANDARD ADJ. HT 12"-16.5"	STANDARD ADJ. HT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"-12"	AS102627	AS102649		AS102693	AS102715	
11"-13"	AS102628	AS102650		AS102694	AS102716	
12"-14"	AS102629	AS102651	AS102673	AS102695	AS102717	AS102739
13"-15"	AS102630	AS102652	AS102674	AS102696	AS102718	AS102740

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-12" CHAIR WIDTH	11"-13" CHAIR WIDTH	12"-14" CHAIR WIDTH	13"-15" CHAIR WIDTH
1" WIDE	CP102246	CP102246	CP101517	CP101517
2" WIDE	CP102232	CP102232	CP100492	CP100492

*NOTE: FOR ITEM 4

8.5"-11.5" (10.5"-12.5" FOR PUSH) BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16.5" (13.5"-16.5" FOR PUSH) BACK HEIGHTS USE TWO 2" BACK STRAPS

16.5"-20.5" (17.5"-20.5" FOR PUSH) BACK HEIGHTS USE THREE 2" BACK STRAPS



BACK UPHOLSTERY (CONT.)

VELCRO ADJUSTABLE FOR TiLite™ TWIST

TABLE THREE - BACKREST EARS


CHAIR WIDTH	STANDARD ADJ. HEIGHT 8.5"-11.5"	STANDARD ADJ. HEIGHT 12"-16.5"	STANDARD ADJ. HEIGHT 16"-20.5"	ADJ. HT 10"-12.5" W/PUSH HANDLES	ADJ. HT 12"-16.5" W/PUSH HANDLES	ADJ. HT 16"-20.5" W/PUSH HANDLES
10"-12" RIGHT	CP102238	CP102241		CP102239	CP102243	
10"-12" LEFT	CP102237	CP102242		CP102240	CP102244	
11"-13" RIGHT	CP102238	CP102241		CP102239	CP102243	
11"-13" LEFT	CP102237	CP102242		CP102240	CP102244	
12"-14" RIGHT	CP101510	CP100450	CP100450	CP101484	CP100508	CP100508
12"-14" LEFT	CP101509	CP100514	CP100514	CP101485	CP100673	CP100673
13"-15" RIGHT	CP101510	CP100450	CP100450	CP101484	CP100508	CP100508
13"-15" LEFT	CP101509	CP100514	CP100514	CP101485	CP100673	CP100673

TABLE FOUR - BACK PAD VELCRO ADJUSTABLE

CHAIR WIDTH	8.5"-11.5" BACK HEIGHT	12"-16.5" BACK HEIGHT	16"-20.5" BACK HEIGHT
	10"-12.5" BACK HEIGHT W/ PUSH	12"-16.5" BACK HEIGHT W/ PUSH	16"-20.5" BACK HEIGHT W/ PUSH
10"-12"	CP102166	CP102188	
11"-13"	CP102167	CP102189	
12"-14"	CP102168	CP102190	CP102212
13"-15"	CP102169	CP102191	CP102213

BACK UPHOLSTERY TENSION ADJ.

TiLite™ YG, YGS

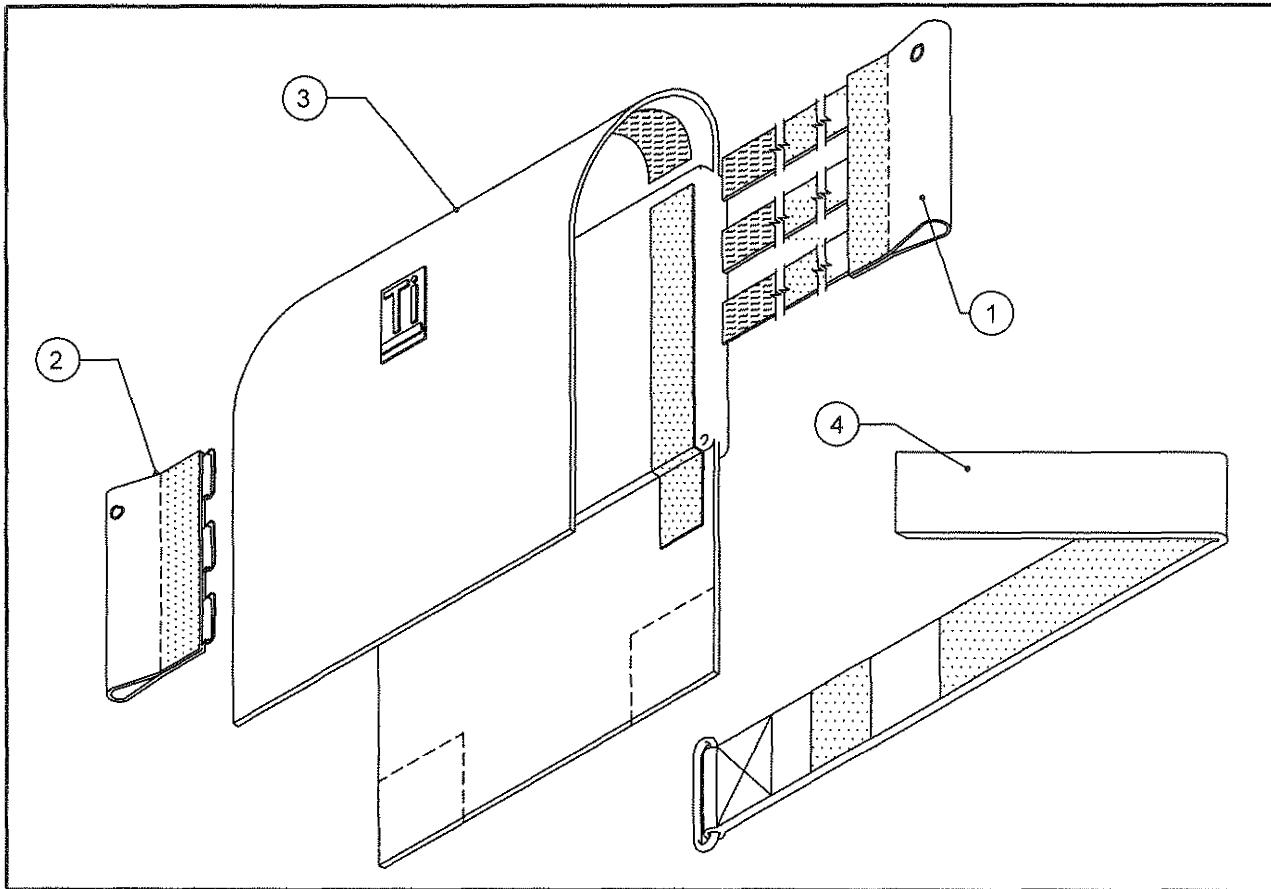

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
10"	AS102616	AS102638	AS102660	AS102682	AS102704	AS102726
11"	AS102617	AS102639	AS102661	AS102683	AS102705	AS102727
12"	AS102618	AS102640	AS102662	AS102684	AS102706	AS102728
13"	AS102619	AS102641	AS102663	AS102685	AS102707	AS102729
14"	AS102620	AS102642	AS102664	AS102686	AS102708	AS102730
15"	AS102621	AS102643	AS102665	AS102687	AS102709	AS102731
16"	AS102622	AS102644	AS102666	AS102688	AS102710	AS102732
17"	AS102623	AS102645	AS102667	AS102689	AS102711	AS102733
18"	AS102624	AS102646	AS102668	AS102690	AS102712	AS102734
19"	AS102625	AS102647	AS102669	AS102691	AS102713	AS102735
20"	AS102626	AS102648	AS102670	AS102692	AS102714	AS102736

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4
 8"-12" BACK HEIGHTS USE TWO 1" BACK STRAPS
 12"-16" BACK HEIGHTS USE ONE 1" AND ONE 2" BACK STRAPS
 16"-20" BACK HEIGHTS USE THREE 2" BACK STRAPS



BACK UPHOLSTERY TENSION ADJ. (CONT)

TiLite™ YG, YGS, DEPTH ADJUSTABLE ZRA (SERIES 1 & 2),
DEPTH ADJUSTABLE AEROS Z (SERIES 1 & 2)

TABLE THREE - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
10"-11" RIGHT	CP102233	CP102234	CP102234	CP102235	CP102236	CP102236
10"-11" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
12"-13" RIGHT	CP101502	CP100049	CP100049	CP101496	CP100323	CP100323
12"-13" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
14"-15" RIGHT	CP101504	CP100264	CP100264	CP101497	CP100324	CP100324
14"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
16"-17" RIGHT	CP101505	CP100265	CP100265	CP101498	CP100325	CP100325
16"-17" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
18"-19" RIGHT	CP101506	CP100266	CP100266	CP101499	CP100326	CP100326
18"-19" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
20" RIGHT	CP101507	CP101508	CP101508	CP101500	CP101494	CP101494
20" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	8"-12" SEAT BACK HEIGHT	12"-16" SEAT BACK HEIGHT	16"-20" SEAT BACK HEIGHT
10"	CP102155	CP102177	CP102199
11"	CP102156	CP102178	CP102200
12"	CP102157	CP102179	CP102201
13"	CP102158	CP102180	CP102202
14"	CP102159	CP102181	CP102203
15"	CP102160	CP102182	CP102204
16"	CP102161	CP102183	CP102205
17"	CP102162	CP102184	CP102206
18"	CP102163	CP102185	CP102207
19"	CP102164	CP102186	CP102208
20"	CP102165	CP102187	CP102209



BREATHABLE BACK UPHOLSTERY TENSION ADJ.

TiLite™ DEPTH ADJUSTABLE ZRA (SERIES 1 & SERIES 2),
DEPTH ADJUSTABLE AERO Z (SERIES 1 & SERIES 2)

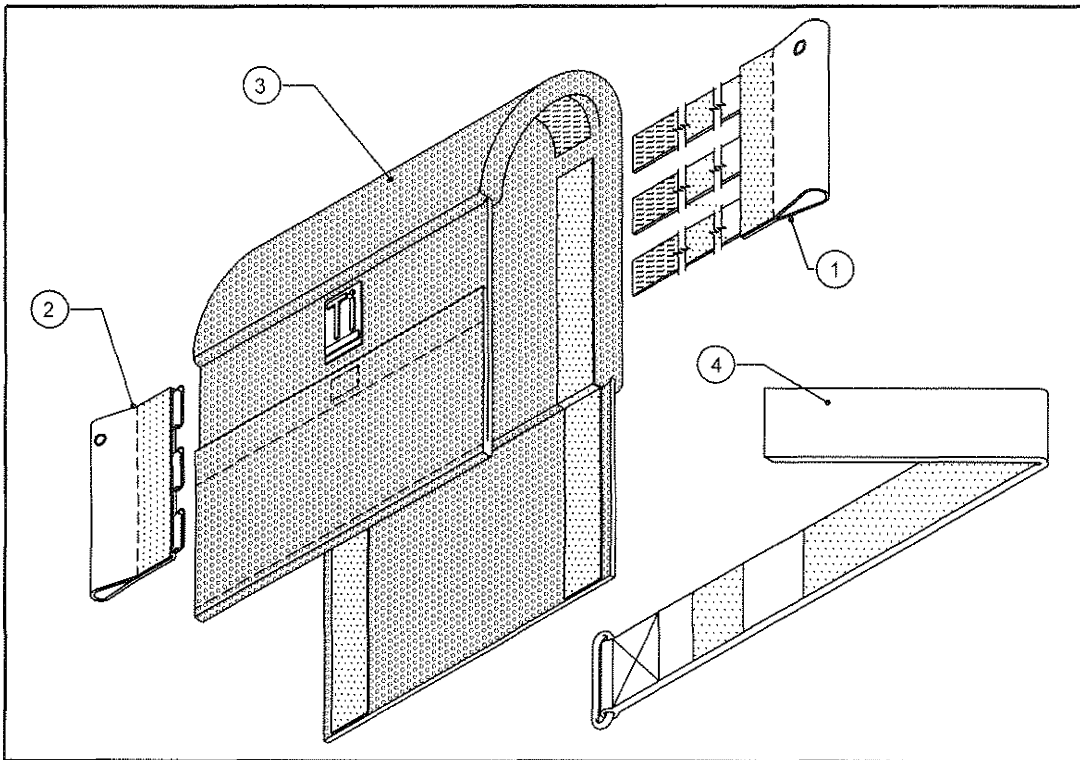

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY W/ PUSH ASSEMBLY
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
10"	AS115809	AS115820	AS115831	AS115842	AS115853	AS115864
11"	AS115810	AS115821	AS115832	AS115843	AS115854	AS115865
12"	AS115811	AS115822	AS115833	AS115844	AS115855	AS115866
13"	AS115812	AS115823	AS115834	AS115845	AS115856	AS115867
14"	AS115813	AS115824	AS115835	AS115846	AS115857	AS115868
15"	AS115814	AS115825	AS115836	AS115847	AS115858	AS115869
16"	AS115815	AS115826	AS115837	AS115848	AS115859	AS115870
17"	AS115816	AS115827	AS115838	AS115849	AS115860	AS115871
18"	AS115817	AS115828	AS115839	AS115850	AS115861	AS115872
19"	AS115818	AS115829	AS115840	AS115851	AS115862	AS115873
20"	AS115819	AS115830	AS115841	AS115852	AS115863	AS115874

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8"-12" BACK HEIGHTS USE TWO 1" BACK STRAPS

12"-16" BACK HEIGHTS USE ONE 1" AND ONE 2" BACK STRAPS

16"-20" BACK HEIGHTS USE THREE 2" BACK STRAPS



BREATHABLE BACK UPHOLSTERY TENSION ADJ. (CONT)

TiLite™ DEPTH ADJUSTABLE ZRA (SERIES 1 & SERIES 2), DEPTH ADJUSTABLE
AERO Z (SERIES 1 & SERIES 2)

TABLE THREE - BACKREST EARS


CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
10"-11" RIGHT	CP102233	CP102234	CP102234	CP102235	CP102236	CP102236
10"-11" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
12"-13" RIGHT	CP101502	CP100049	CP100049	CP101496	CP100323	CP100323
12"-13" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
14"-15" RIGHT	CP101504	CP100264	CP100264	CP101497	CP100324	CP100324
14"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
16"-17" RIGHT	CP101505	CP100265	CP100265	CP101498	CP100325	CP100325
16"-17" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
18"-19" RIGHT	CP101506	CP100266	CP100266	CP101499	CP100326	CP100326
18"-19" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
20" RIGHT	CP101507	CP101508	CP101508	CP101500	CP101494	CP101494
20" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	8"-12" SEAT BACK HEIGHT	12"-16" SEAT BACK HEIGHT	16"-20" SEAT BACK HEIGHT
10"	CP107086	CP107097	CP107108
11"	CP107087	CP107098	CP107109
12"	CP107088	CP107099	CP107110
13"	CP107089	CP107100	CP107111
14"	CP107090	CP107101	CP107112
15"	CP107091	CP107102	CP107113
16"	CP107092	CP107103	CP107114
17"	CP107093	CP107104	CP107115
18"	CP107094	CP107105	CP107116
19"	CP107095	CP107106	CP107117
20"	CP107096	CP107107	CP107118

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES TiLite™ YG, YGS, DEPTH ADJUSTABLE ZRA (SERIES 1 & 2), DEPTH ADJUSTABLE AERO Z (SERIES 1 & 2)

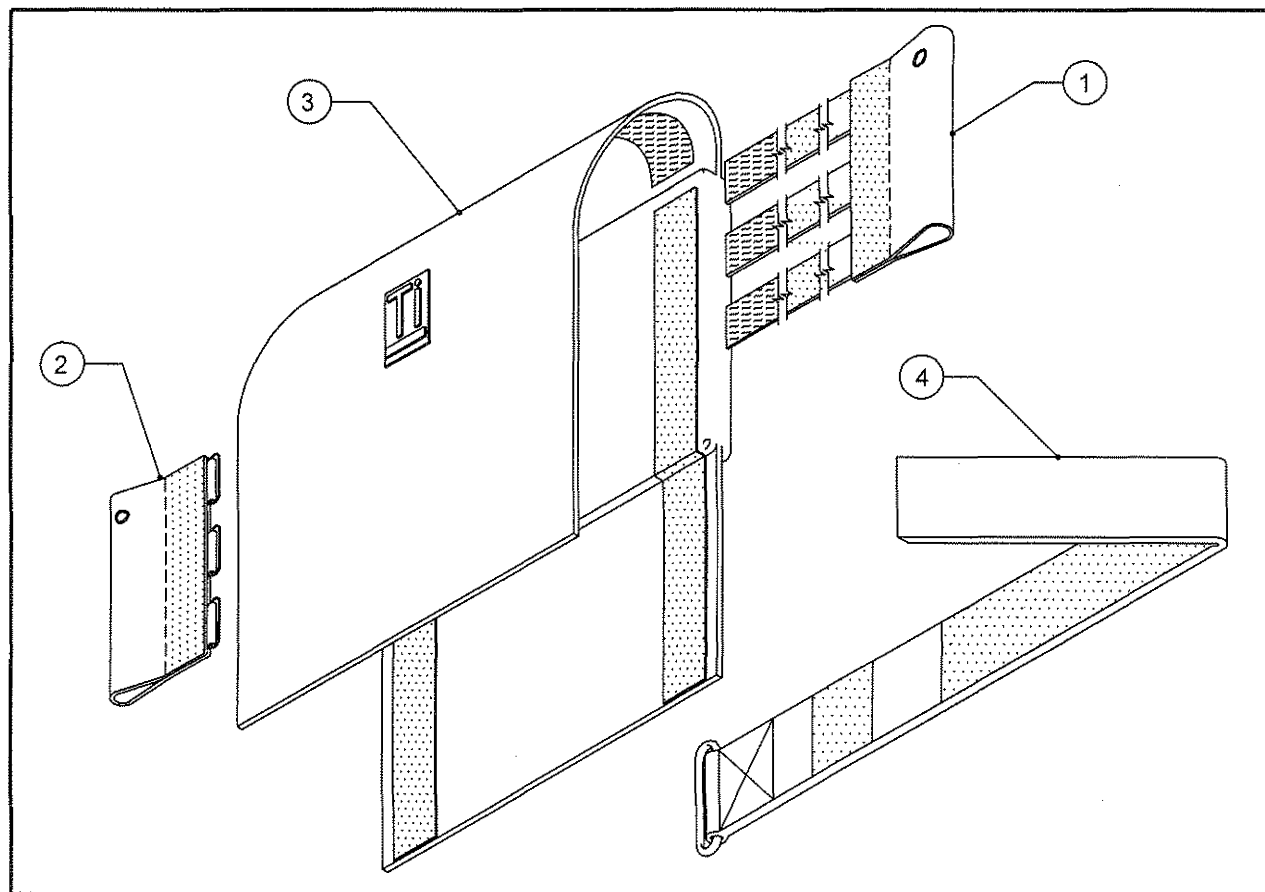

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 12"-14"	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 18"-20.5"
10"	AS104034	AS104045	AS104056
11"	AS104035	AS104046	AS104057
12"	AS104036	AS104047	AS104058
13"	AS104037	AS104048	AS104059
14"	AS104038	AS104049	AS104060
15"	AS104039	AS104050	AS104061
16"	AS104040	AS104051	AS104062
17"	AS104041	AS104052	AS104063
18"	AS104042	AS104053	AS104064
19"	AS104043	AS104054	AS104065
20"	AS104044	AS104055	AS104066

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

12"-14" BACK HEIGHTS USE TWO 1" BACK STRAPS

14"-18.5" BACK HEIGHTS USE ONE 1" AND ONE 2" BACK STRAPS

18"-20.5" BACK HEIGHTS USE THREE 2" BACK STRAPS



TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES TiLite™ YG, YGS
DEPTH ADJUSTABLE ZRA (SERIES 1 & 2), DEPTH ADJUSTABLE AERO Z (SERIES 1 & 2)

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 12"-14"	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 18"-20.5"
10"-11" RIGHT	CP103685	CP103716	CP103716
10"-11" LEFT	CP103684	CP103715	CP103715
12"-13" RIGHT	CP103686	CP103717	CP103717
12"-13" LEFT	CP103684	CP103715	CP103715
14"-15" RIGHT	CP103687	CP103718	CP103718
14"-15" LEFT	CP103684	CP103715	CP103715
16"-17" RIGHT	CP103688	CP103719	CP103719
16"-17" LEFT	CP103684	CP103715	CP103715
18"-19" RIGHT	CP103689	CP103720	CP103720
18"-19" LEFT	CP103684	CP103715	CP103715
20" RIGHT	CP103690	CP103721	CP103721
20" LEFT	CP103684	CP103715	CP103715


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	12"-14" SEAT BACK HEIGHT	14"-18.5" SEAT BACK HEIGHT	18"-20.5" SEAT BACK HEIGHT
10"	CP102155	CP102177	CP102199
11"	CP102156	CP102178	CP102200
12"	CP102157	CP102179	CP102201
13"	CP102158	CP102180	CP102202
14"	CP102159	CP102181	CP102203
15"	CP102160	CP102182	CP102204
16"	CP102161	CP102183	CP102205
17"	CP102162	CP102184	CP102206
18"	CP102163	CP102185	CP102207
19"	CP102164	CP102186	CP102208
20"	CP102165	CP102187	CP102209



BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES TiLite™ DEPTH ADJUSTABLE AERO Z
(SERIES 1 & SERIES 2) AND DEPTH ADJUSTABLE ZRA (SERIES 1 & SERIES 2)

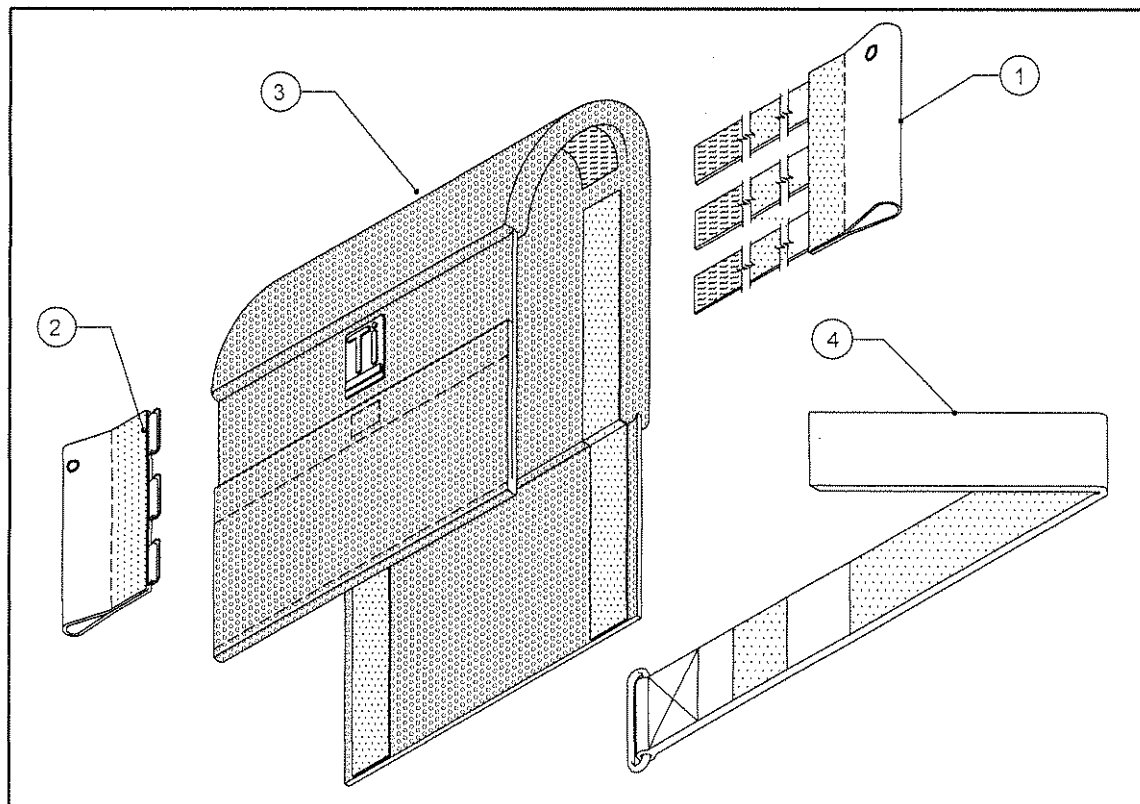

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 12"-14"	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 18"-20.5"
10"	AS115875	AS115886	AS115897
11"	AS115876	AS115887	AS115898
12"	AS115877	AS115888	AS115899
13"	AS115878	AS115889	AS115900
14"	AS115879	AS115890	AS115901
15"	AS115880	AS115891	AS115902
16"	AS115881	AS115892	AS115903
17"	AS115882	AS115893	AS115904
18"	AS115883	AS115894	AS115905
19"	AS115884	AS115895	AS115906
20"	AS115885	AS115896	AS115907

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

12"-14" BACK HEIGHTS USE TWO 1" BACK STRAPS

14"-18.5" BACK HEIGHTS USE ONE 1" AND ONE 2" BACK STRAPS

18"-20.5" BACK HEIGHTS USE THREE 2" BACK STRAPS



BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES TiLite™ DEPTH ADJUSTABLE AERO Z
(SERIES 1 & SERIES 2) AND DEPTH ADJUSTABLE ZRA (SERIES 1 & SERIES 2)

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 12"-14"	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 18"-20.5"
10"-11" RIGHT	CP103685	CP103716	CP103716
10"-11" LEFT	CP103684	CP103715	CP103715
12"-13" RIGHT	CP103686	CP103717	CP103717
12"-13" LEFT	CP103684	CP103715	CP103715
14"-15" RIGHT	CP103687	CP103718	CP103718
14"-15" LEFT	CP103684	CP103715	CP103715
16"-17" RIGHT	CP103688	CP103719	CP103719
16"-17" LEFT	CP103684	CP103715	CP103715
18"-19" RIGHT	CP103689	CP103720	CP103720
18"-19" LEFT	CP103684	CP103715	CP103715
20" RIGHT	CP103690	CP103721	CP103721
20" LEFT	CP103684	CP103715	CP103715


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	12"-14" SEAT BACK HEIGHT	14"-18.5" SEAT BACK HEIGHT	18"-20.5" SEAT BACK HEIGHT
10"	CP107086	CP107097	CP107108
11"	CP107087	CP107098	CP107109
12"	CP107088	CP107099	CP107110
13"	CP107089	CP107100	CP107111
14"	CP107090	CP107101	CP107112
15"	CP107091	CP107102	CP107113
16"	CP107092	CP107103	CP107114
17"	CP107093	CP107104	CP107115
18"	CP107094	CP107105	CP107116
19"	CP107095	CP107106	CP107117
20"	CP107096	CP107107	CP107118



BACK UPHOLSTERY VELCRO ADJ.

TiLite™ YG, YGS

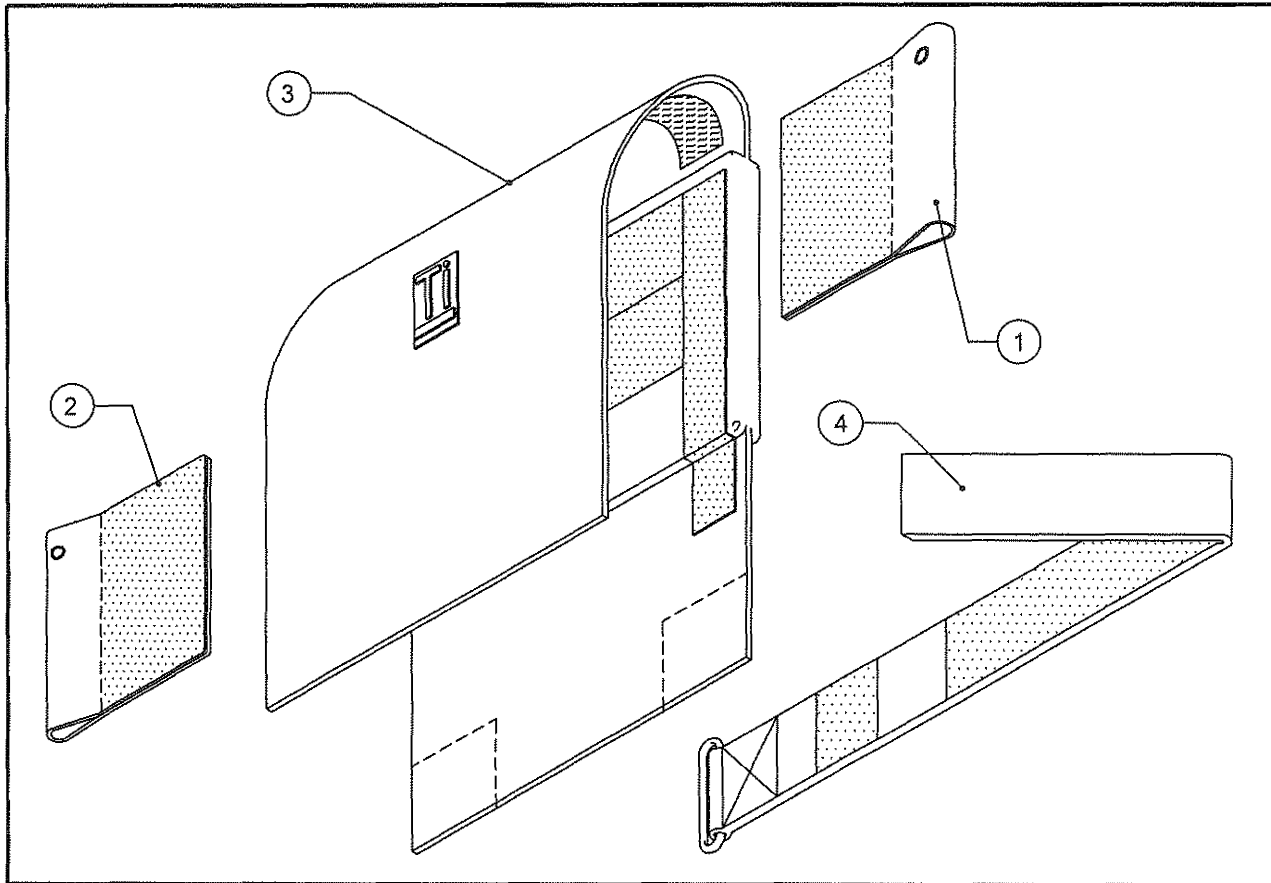

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD VELCRO ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4**		TABLE ONE	VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - VELCRO ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
10"	AS102627	AS102649	AS102671	AS102693	AS102715	AS102737
11"	AS102628	AS102650	AS102672	AS102694	AS102716	AS102738
12"	AS102629	AS102651	AS102673	AS102695	AS102717	AS102739
13"	AS102630	AS102652	AS102674	AS102696	AS102718	AS102740
14"	AS102631	AS102653	AS102675	AS102697	AS102719	AS102741
15"	AS102632	AS102654	AS102676	AS102698	AS102720	AS102742
16"	AS102633	AS102655	AS102677	AS102699	AS102721	AS102743
17"	AS102634	AS102656	AS102678	AS102700	AS102722	AS102744
18"	AS102635	AS102657	AS102679	AS102701	AS102723	AS102745
19"	AS102636	AS102658	AS102680	AS102702	AS102724	AS102746
20"	AS102637	AS102659	AS102681	AS102703	AS102725	AS102747

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	10"-11" CHAIR WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP102246	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP102232	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8"-12" BACK HEIGHTS USE TWO 1" BACK STRAPS

12"-16" BACK HEIGHTS USE ONE 1" AND ONE 2" BACK STRAPS

16"-20" BACK HEIGHTS USE THREE 2" BACK STRAPS



BACK UPHOLSTERY VELCRO ADJ. (CONT)

TiLite™ YG, YGS, DEPTH ADJUSTABLE ZRA (SERIES 1 & 2),
DEPTH ADJUSTABLE AERO Z (SERIES 1 & 2)

TABLE THREE - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
10"-11" RIGHT	CP102238	CP102241	CP102241	CP102239	CP102243	CP102243
10"-11" LEFT	CP102237	CP102242	CP102242	CP102240	CP102244	CP102244
12"-13" RIGHT	CP101510	CP100450	CP100450	CP101484	CP100508	CP100508
12"-13" LEFT	CP101509	CP100514	CP100514	CP101485	CP100673	CP100673
14"-18" RIGHT	CP101511	CP100451	CP100451	CP101486	CP100509	CP100509
14"-18" LEFT	CP101512	CP100454	CP100454	CP101487	CP100507	CP100507
19"-20" RIGHT	CP101524	CP101526	CP101526	CP101530	CP101528	CP101528
19"-20" LEFT	CP101523	CP101525	CP101525	CP101529	CP101527	CP101527


TABLE FOUR - BACK PAD VELCRO ADJUSTABLE

CHAIR WIDTH	8"-12" SEAT BACK HEIGHT	12"-16" SEAT BACK HEIGHT	16"-20" SEAT BACK HEIGHT
10"	CP102166	CP102188	CP102210
11"	CP102167	CP102189	CP102211
12"	CP102168	CP102190	CP102212
13"	CP102169	CP102191	CP102213
14"	CP102170	CP102192	CP102214
15"	CP102171	CP102193	CP102215
16"	CP102172	CP102194	CP102216
17"	CP102173	CP102195	CP102217
18"	CP102174	CP102196	CP102218
19"	CP102175	CP102197	CP102219
20"	CP102176	CP102198	CP102220



BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ TR, TRC, EVO, TRA, BB, TRA, YR, YRA

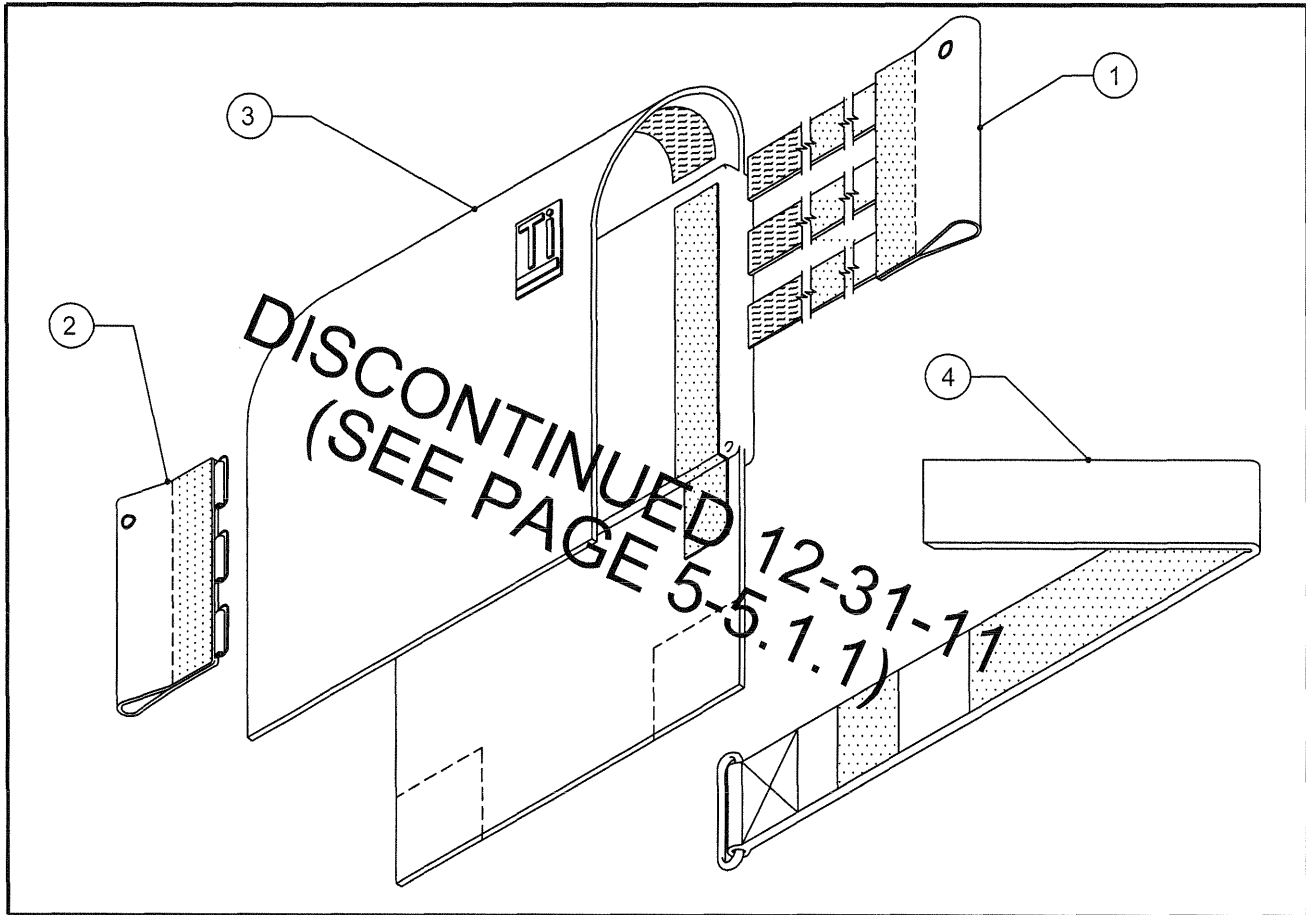

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"	AS101007	AS100132	AS100146	AS101014	AS100167	AS100181
13"	AS101008	AS100133	AS100147	AS101015	AS100168	AS100182
14"	AS101009	AS100134	AS100148	AS101016	AS100169	AS100183
15"	AS101010	AS100135	AS100149	AS101017	AS100170	AS100184
16"	AS101011	AS100136	AS100150	AS101018	AS100171	AS100185
17"	AS101012	AS100137	AS100151	AS101019	AS100172	AS100186
18"	AS101013	AS100138	AS100152	AS101020	AS100173	AS100187
19"	AS101554	AS101556	AS101560	AS101562	AS101564	AS101566
20"	AS101555	AS101557	AS101561	AS101563	AS101565	AS101567

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8"-12" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16" BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20" BACK HEIGHTS USE THREE 2" BACK STRAPS



Parts Manual

BACK UPHOLSTERY (CONT.)

TENSION ADJUSTABLE FOR TiLite™ TR, TRC, EVO, TRA, BB, TRA, YR, YRA

TABLE THREE - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"-13" RIGHT	CP101502	CP100049	CP100049	CP101496	CP100323	CP100323
12"-13" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
14"-15" RIGHT	CP101504	CP100264	CP100264	CP101497	CP100324	CP100324
14"-15" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
16"-17" RIGHT	CP101505	CP100265	CP100265	CP101498	CP100325	CP100325
16"-17" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
18"-19" RIGHT	CP101506	CP100266	CP100266	CP101499	CP100326	CP100326
18"-19" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322
20" RIGHT	CP101507	CP101508	CP101508	CP101500	CP101494	CP101494
20" LEFT	CP101503	CP100050	CP100050	CP101495	CP100322	CP100322

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	8"-12" SEAT BACK HEIGHT	12"-16" SEAT BACK HEIGHT	16"-20" SEAT BACK HEIGHT
12"	CP101253	CP100098	CP100883
13"	CP101254	CP100099	CP100884
14"	CP101255	CP100100	CP100885
15"	CP101256	CP100101	CP100886
16"	CP101257	CP100102	CP100887
17"	CP101258	CP100103	CP100888
18"	CP101259	CP100104	CP100889
19"	CP101532	CP101536	CP101488
20"	CP101533	CP101537	CP101489


DISCONTINUED 12-31-11
(SEE PAGE 5-5.1.2)



TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES

TiLite™ TR, TRC, EVO, TRA, YR, YRA

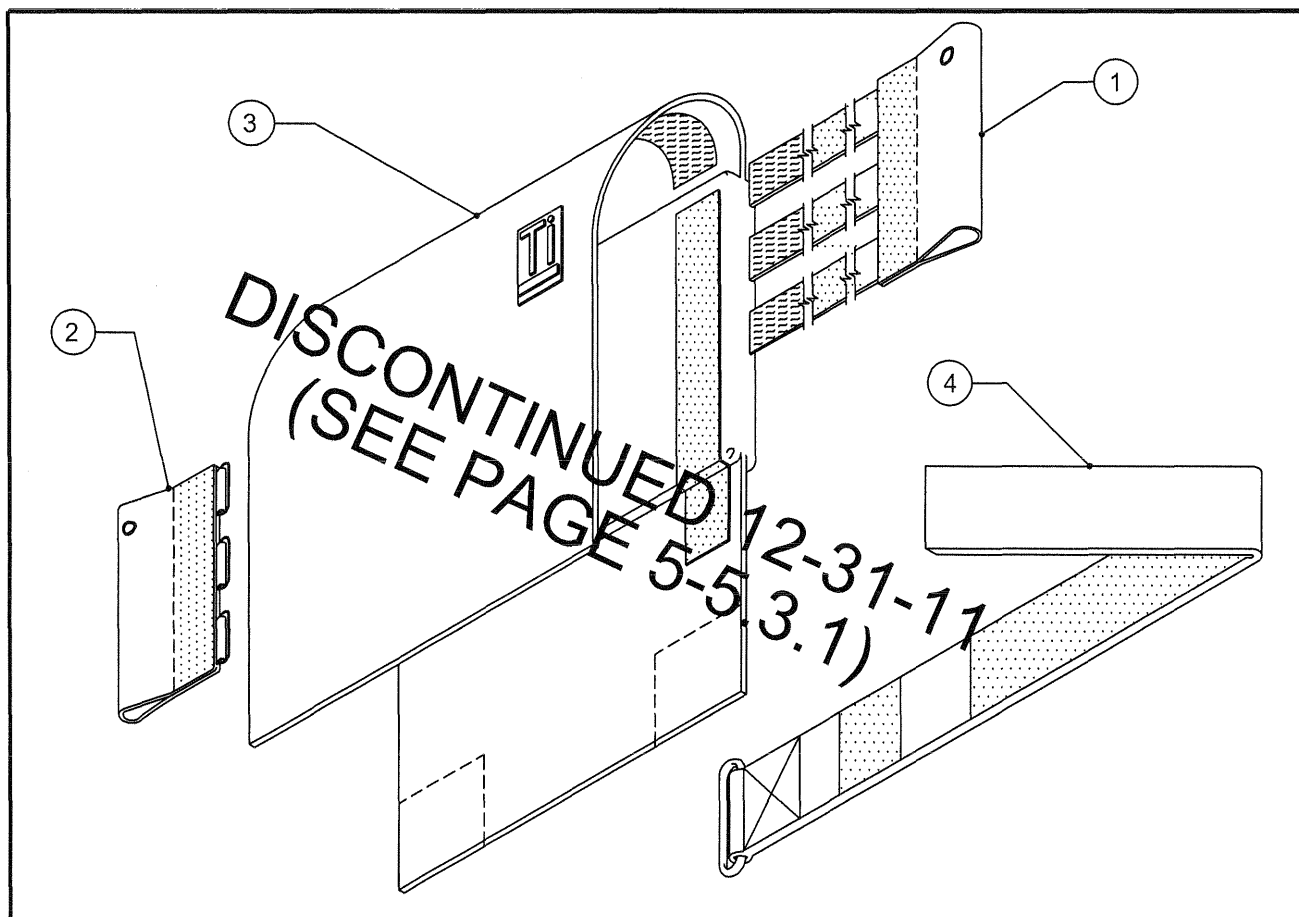

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 12"-14"	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 18"-20.5"
12"	AS104003	AS104014	AS104025
13"	AS104004	AS104015	AS104026
14"	AS104005	AS104016	AS104027
15"	AS104006	AS104017	AS104028
16"	AS104007	AS104018	AS104029
17"	AS104008	AS104019	AS104030
18"	AS104009	AS104020	AS104031
19"	AS104010	AS104021	AS104032
20"	AS104011	AS104022	AS104033

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

12"-14" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

14"-18.5" BACK HEIGHTS USE TWO 2" BACK STRAPS

18"-20.5" BACK HEIGHTS USE THREE 2" BACK STRAPS

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ TR, TRC, EVO, TRA, YR, YRA

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 12"-14"	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 18"-20.5"
12"-13" RIGHT	CP103686	CP103717	CP103717
12"-13" LEFT	CP103684	CP103715	CP103715
14"-15" RIGHT	CP103687	CP103718	CP103718
14"-15" LEFT	CP103684	CP103715	CP103715
16"-17" RIGHT	CP103688	CP103719	CP103719
16"-17" LEFT	CP103684	CP103715	CP103715
18"-19" RIGHT	CP103689	CP103720	CP103720
18"-19" LEFT	CP103684	CP103715	CP103715
20" RIGHT	CP103690	CP103721	CP103721
20" LEFT	CP103684	CP103715	CP103715

TABLE FOUR - BACK PAD TENSION ADJUSTABLE


CHAIR WIDTH	ADJ. HEIGHT 12"-14"	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 18"-20.5"
12"	CP101253	CP100098	CP100883
13"	CP101254	CP100099	CP100884
14"	CP101255	CP100100	CP100885
15"	CP101256	CP100101	CP100886
16"	CP101257	CP100102	CP100887
17"	CP101258	CP100103	CP100888
18"	CP101259	CP100104	CP100889
19"	CP101532	CP101536	CP101488
20"	CP101533	CP101537	CP101489

DISCONTINUED 12-31-11
(SEE PAGE 5-5.3.2)



BACK UPHOLSTERY

VELCRO ADJUSTABLE FOR TiLite™ TR, TRC, EVO, TRA, BB, TRA, YR, YRA

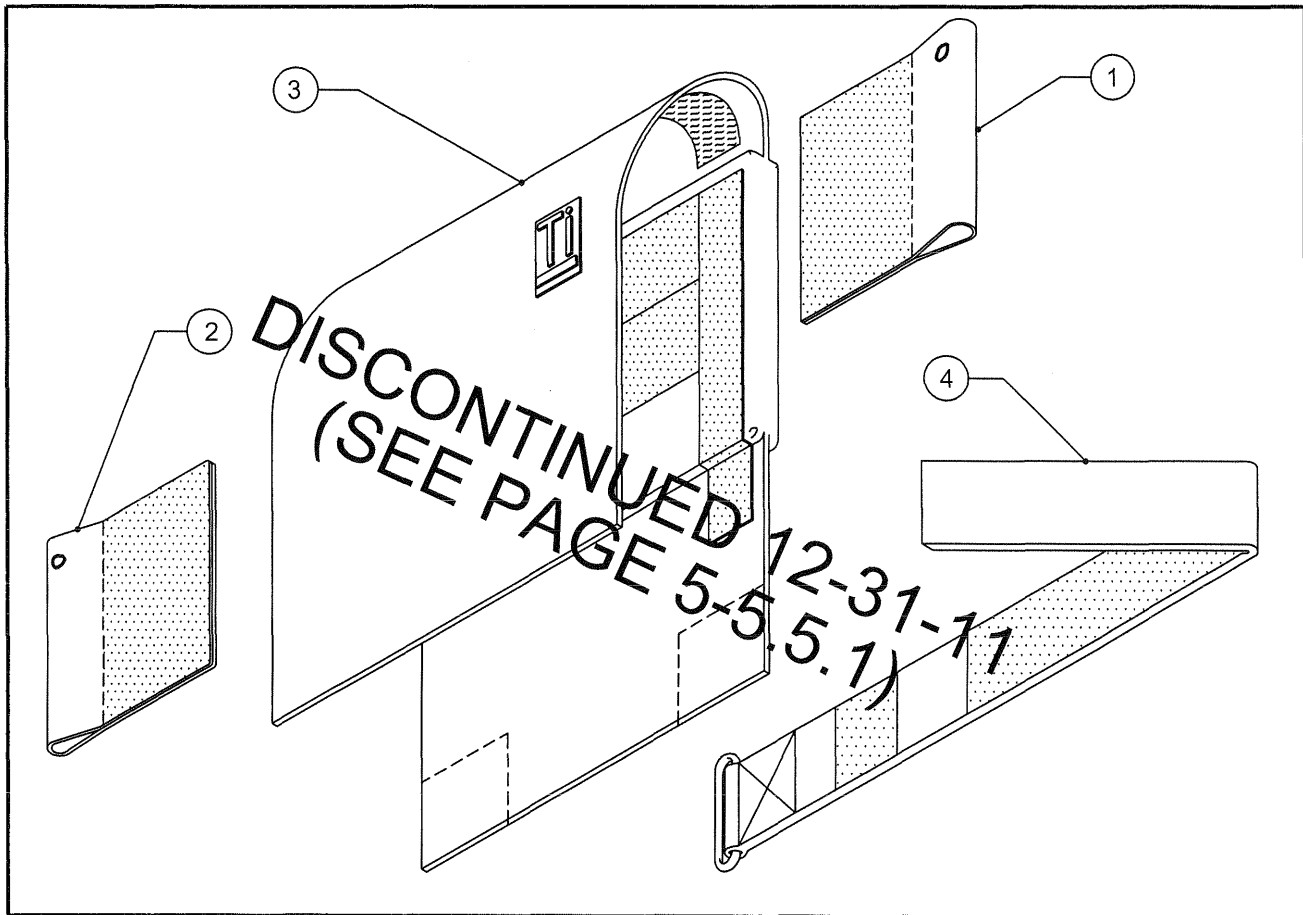

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4		TABLE ONE	VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"	AS100979	AS100125	AS100993	AS100986	AS100160	AS101000
13"	AS100980	AS100126	AS100994	AS100987	AS100161	AS101001
14"	AS100981	AS100127	AS100995	AS100988	AS100162	AS101002
15"	AS100982	AS100128	AS100996	AS100989	AS100163	AS101003
16"	AS100983	AS100129	AS100997	AS100990	AS100164	AS101004
17"	AS100984	AS100130	AS100998	AS100991	AS100165	AS101005
18"	AS100985	AS100131	AS100999	AS100992	AS100166	AS101006
19"	AS101533	AS101535	AS101546	AS101548	AS101550	AS101552
20"	AS101534	AS101536	AS101547	AS101549	AS101551	AS101553

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

8"-12" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP

12"-16" BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20" BACK HEIGHTS USE THREE 2" BACK STRAPS



BACK UPHOLSTERY (CONT.)

VELCRO ADJUSTABLE FOR TiLite™ TR, TRC, EVO, TRA, BB, TRA, YR, YRA

TABLE THREE - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HEIGHT 8"-12"	STANDARD ADJ. HEIGHT 12"-16"	STANDARD ADJ. HEIGHT 16"-20"	ADJ. HT 10"-12" W/PUSH HANDLES	ADJ. HT 13"-16" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"-13" RIGHT	CP101510	CP100450	CP100450	CP101484	CP100508	CP100508
12"-13" LEFT	CP101509	CP100514	CP100514	CP101485	CP100673	CP100673
14"-18" RIGHT	CP101511	CP100451	CP100451	CP101486	CP100509	CP100509
14"-18" LEFT	CP101512	CP100454	CP100454	CP101487	CP100507	CP100507
19"-20" RIGHT	CP101524	CP101526	CP101526	CP101530	CP101528	CP101528
19"-20" LEFT	CP101523	CP101525	CP101525	CP101529	CP101527	CP101527


TABLE FOUR - BACK PAD VELCRO ADJUSTABLE

CHAIR WIDTH	8"-12" SEAT BACK HEIGHT	12"-16" SEAT BACK HEIGHT	16"-20" SEAT BACK HEIGHT
12"	CP101205	CP100460	CP100890
13"	CP101206	CP100461	CP100891
14"	CP101207	CP100462	CP100892
15"	CP101208	CP100463	CP100893
16"	CP101209	CP100464	CP100894
17"	CP101210	CP100465	CP100895
18"	CP101211	CP100466	CP100896
19"	CP101534	CP101538	CP101540
20"	CP101535	CP101539	CP101541

DISCONTINUED 12-31-11
(SEE PAGE 5-5.5.2)

TENSION ADJ. BACK UPHOLSTERY

TiLite™ X, SX

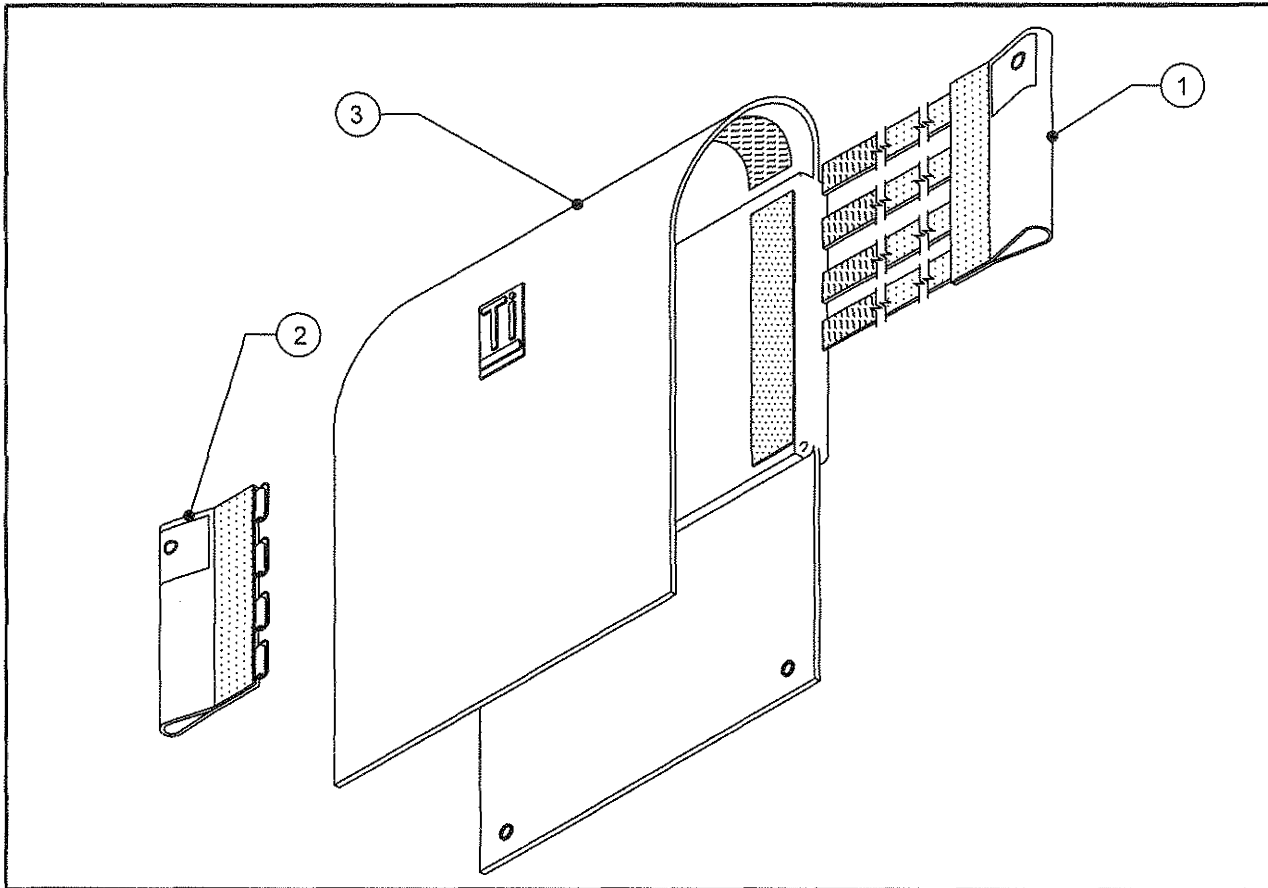

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-3		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE- TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLY

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT X, SX 11"-14"	ADJ. HEIGHT X, SX 15"-18"	ADJ. HEIGHT X, SX 11"-14"	ADJ. HEIGHT X, SX 15"-18"
12"	AS100422	AS100429	AS100279	AS100286
13"	AS100423	AS100430	AS100280	AS100287
14"	AS100424	AS100431	AS100281	AS100288
15"	AS100425	AS100432	AS100282	AS100289
16"	AS100426	AS100433	AS100283	AS100290
17"	AS100427	AS100434	AS100284	AS100291
18"	AS100428	AS100435	AS100285	AS100292
19"	AS101570	AS101572	AS101574	AS101576
20"	AS101571	AS101573	AS101575	AS101577

BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ X, SX

TABLE TWO

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT X, SX 11"-14"	ADJ. HEIGHT X, SX 15"-18"	ADJ. HEIGHT X, SX 11"-14"	ADJ. HEIGHT X, SX 15"-18"
12"-13"	CP100808	CP100814	CP100591	CP100596
14"-15"	CP100809	CP100815	CP100592	CP100597
16"-17"	CP100810	CP100816	CP100593	CP100598
18"-19"	CP100811	CP100817	CP100594	CP100599
20"	CP101551	CP101550	CP101552	CP101549

TABLE THREE


CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT X, SX 11"-14"	ADJ. HEIGHT X, SX 15"-18"	ADJ. HEIGHT X, SX 11"-14"	ADJ. HEIGHT X, SX 15"-18"
ALL WIDTHS	CP100812	CP100818	CP100595	CP100600

TABLE FOUR

CHAIR WIDTH	ADJ. HEIGHT X, SX 11"-14"	ADJ. HEIGHT X, SX 15"-18"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556

TENSION ADJ. BACK UPHOLSTERY

TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

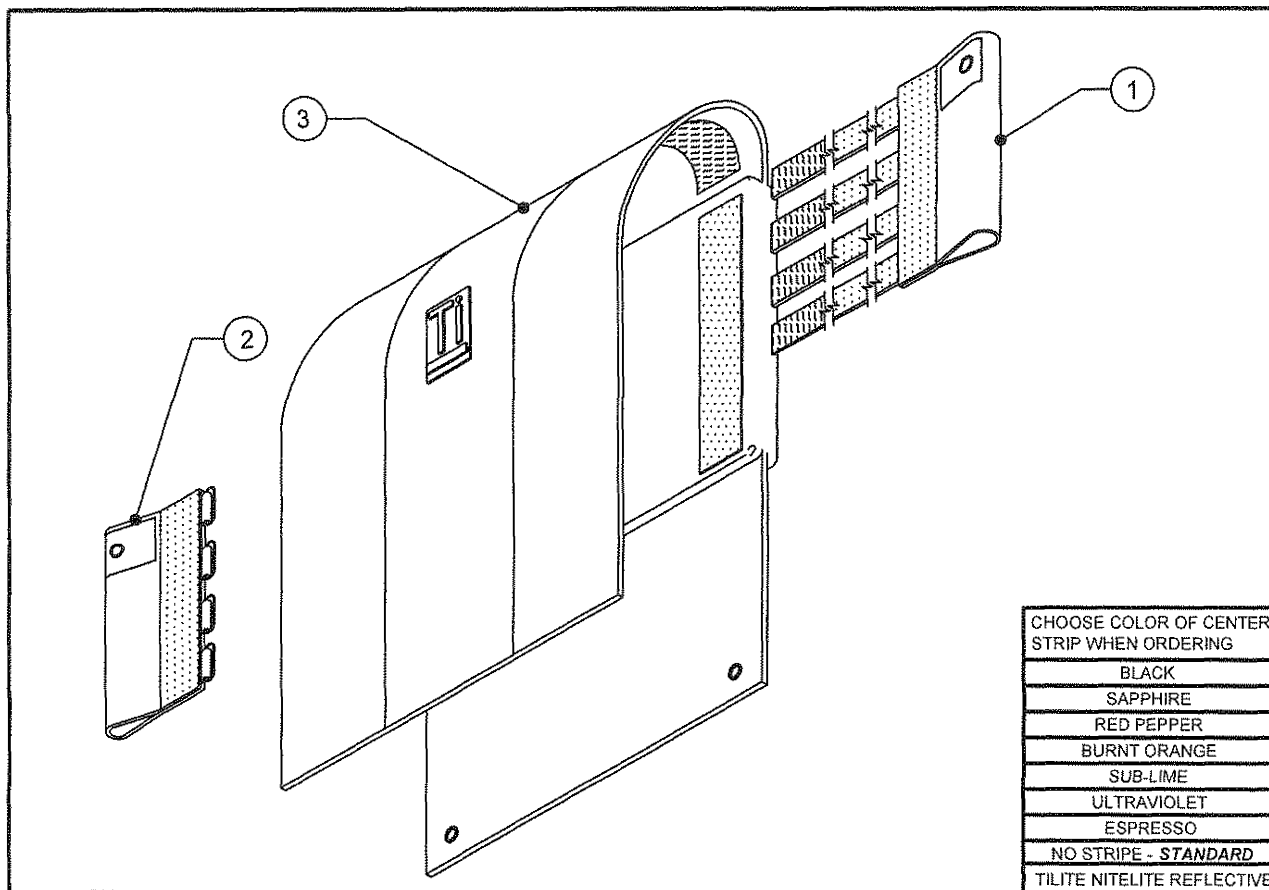

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-3		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE- TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLY (ITEMS 1-3)

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT AERO X,2GX 11"-15"	ADJ. HEIGHT AERO X,2GX 16"-20"	ADJ. HEIGHT AERO X,2GX 11"-15"	ADJ. HEIGHT AERO X,2GX 16"-20"
12"	AS100422	AS100429	AS100279	AS100286
13"	AS100423	AS100430	AS100280	AS100287
14"	AS100424	AS100431	AS100281	AS100288
15"	AS100425	AS100432	AS100282	AS100289
16"	AS100426	AS100433	AS100283	AS100290
17"	AS100427	AS100434	AS100284	AS100291
18"	AS100428	AS100435	AS100285	AS100292
19"	AS101570	AS101572	AS101574	AS101576
20"	AS101571	AS101573	AS101575	AS101577

BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT AERO X,2GX 11"-15"	ADJ. HEIGHT AERO X,2GX 16"-20"	ADJ. HEIGHT AERO X,2GX 11"-15"	ADJ. HEIGHT AERO X,2GX 16"-20"
12"-13"	CP100808	CP100814	CP100591	CP100596
14"-15"	CP100809	CP100815	CP100592	CP100597
16"-17"	CP100810	CP100816	CP100593	CP100598
18"-19"	CP100811	CP100817	CP100594	CP100599
20"	CP101551	CP101550	CP101552	CP101549

TABLE THREE


CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT AERO X,2GX 11"-15"	ADJ. HEIGHT AERO X,2GX 16"-20"	ADJ. HEIGHT AERO X,2GX 11"-15"	ADJ. HEIGHT AERO X,2GX 16"-20"
ALL WIDTHS	CP100812	CP100818	CP100595	CP100600

TABLE FOUR

CHAIR WIDTH	ADJ. HEIGHT AERO X,2GX 11"-15"	ADJ. HEIGHT AERO X,2GX 16"-20"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556

BREATHABLE TENSION ADJ. BACK UPHOLSTERY

TiLite™ AERO X, 2GX, X, SX, Aero X (SERIES 2), 2GX (SERIES 2)

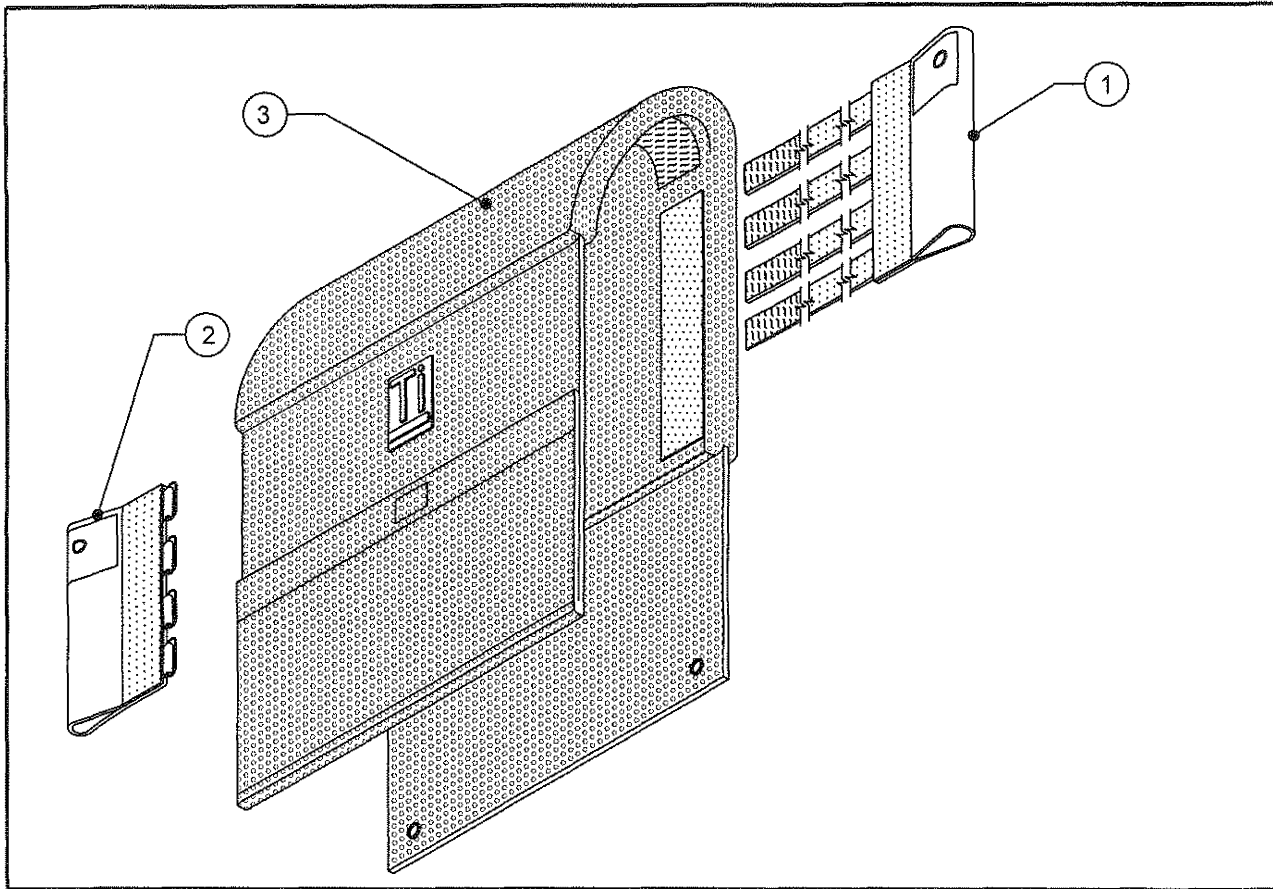

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-3		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY W/ PUSH ASSEMBLY
1-3		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE- TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLY

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 11"-14"	ADJ. HEIGHT 15"-18"	ADJ. HEIGHT 11"-14"	ADJ. HEIGHT 15"-18"
12"	AS115440	AS115449	AS115458	AS115467
13"	AS115441	AS115450	AS115459	AS115468
14"	AS115442	AS115451	AS115460	AS115469
15"	AS115443	AS115452	AS115461	AS115470
16"	AS115444	AS115453	AS115462	AS115471
17"	AS115445	AS115454	AS115463	AS115472
18"	AS115446	AS115455	AS115464	AS115473
19"	AS115447	AS115456	AS115465	AS115474
20"	AS115448	AS115457	AS115466	AS115475

BREATHABLE BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ AERO X, 2GX, X, SX,
Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 11"-14"	ADJ. HEIGHT 15"-18"	ADJ. HEIGHT 11"-14"	ADJ. HEIGHT 15"-18"
12"-13"	CP100808	CP100814	CP100591	CP100596
14"-15"	CP100809	CP100815	CP100592	CP100597
16"-17"	CP100810	CP100816	CP100593	CP100598
18"-19"	CP100811	CP100817	CP100594	CP100599
20"	CP101551	CP101550	CP101552	CP101549

TABLE THREE

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 11"-14"	ADJ. HEIGHT 15"-18"	ADJ. HEIGHT 11"-14"	ADJ. HEIGHT 15"-18"
ALL WIDTHS	CP100812	CP100818	CP100595	CP100600

TABLE FOUR


CHAIR WIDTH	ADJ. HEIGHT 11"-14"	ADJ. HEIGHT 15"-18"
12"	CP107023	CP107032
13"	CP107024	CP107033
14"	CP107025	CP107034
15"	CP107026	CP107035
16"	CP107027	CP107036
17"	CP107028	CP107037
18"	CP107029	CP107038
19"	CP107030	CP107039
20"	CP107031	CP107040



TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES

TiLite™ X, SX

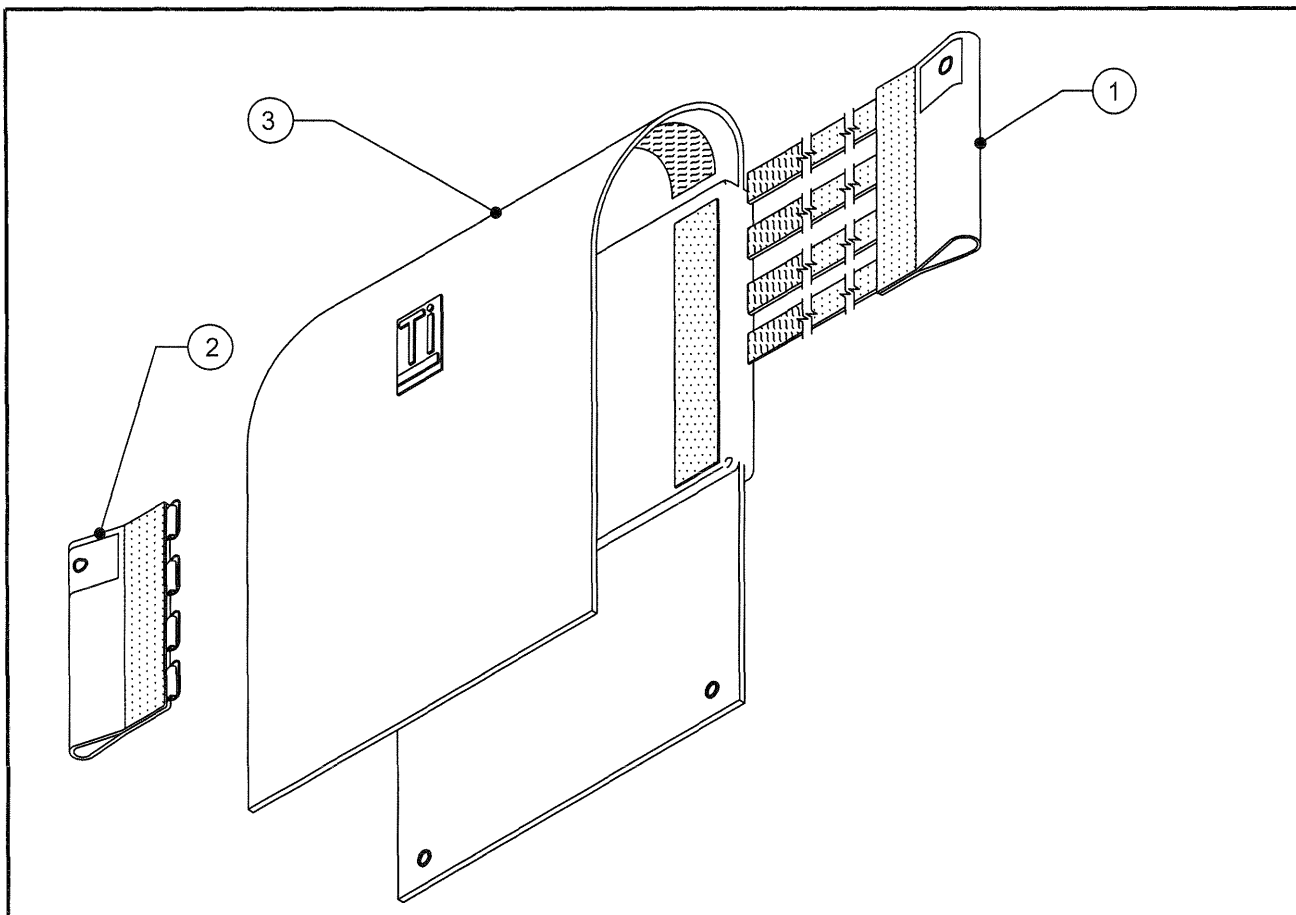

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE TWO	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE THREE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-3		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	X, SX ADJ. HEIGHT 11.5"-14.5"	X,SX ADJ. HEIGHT 15.5"-18.5"
12"	AS104085	AS104094
13"	AS104086	AS104095
14"	AS104087	AS104096
15"	AS104088	AS104097
16"	AS104089	AS104098
17"	AS104090	AS104099
18"	AS104091	AS104100
19"	AS104092	AS104101
20"	AS104093	AS104102

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES

TiLite™ X, SX

TABLE TWO - BACKREST EARS

CHAIR WIDTH	X, SX ADJ. HEIGHT 11.5"-14.5"	X,SX ADJ. HEIGHT 15.5"-18.5"
12"-13" RIGHT	CP103693	CP103700
12"-13" LEFT	CP103691	CP103698
14"-15" RIGHT	CP103694	CP103701
14"-15" LEFT	CP103691	CP103698
16"-17" RIGHT	CP103695	CP103702
16"-17" LEFT	CP103691	CP103698
18"-19" RIGHT	CP103696	CP103703
18"-19" LEFT	CP103691	CP103698
20" RIGHT	CP103697	CP103704
20" LEFT	CP103691	CP103698


TABLE THREE - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	X, SX ADJ. HEIGHT 11.5"-14.5"	X,SX ADJ. HEIGHT 15.5"-18.5"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556



TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

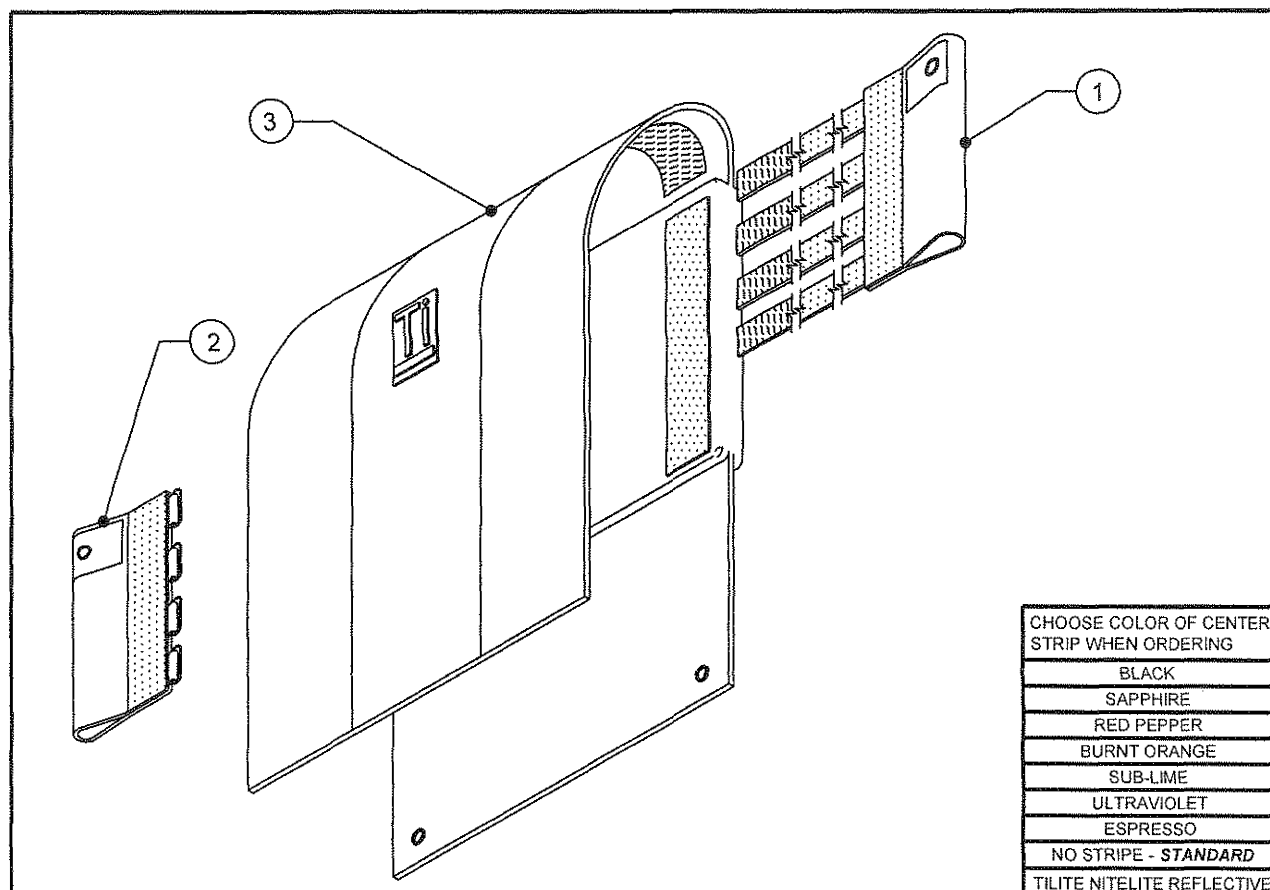

CHOOSE COLOR OF CENTER STRIP WHEN ORDERING
BLACK
SAPPHIRE
RED PEPPER
BURNT ORANGE
SUB-LIME
ULTRAVIOLET
ESPRESSO
NO STRIPE - STANDARD
TILITE NITELITE REFLECTIVE

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE TWO	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE THREE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-3		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	2GX, Aero X ADJ. HEIGHT 11.5"-15.5"	2GX, Aero X ADJ. HEIGHT 16.5"-20.5"
12"	AS104085	AS104094
13"	AS104086	AS104095
14"	AS104087	AS104096
15"	AS104088	AS104097
16"	AS104089	AS104098
17"	AS104090	AS104099
18"	AS104091	AS104100
19"	AS104092	AS104101
20"	AS104093	AS104102

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO - BACKREST EARS

CHAIR WIDTH	2GX, Aero X ADJ. HEIGHT 11.5"-15.5"	2GX, Aero X ADJ. HEIGHT 16.5"-20.5"
12"-13" RIGHT	CP103693	CP103700
12"-13" LEFT	CP103691	CP103698
14"-15" RIGHT	CP103694	CP103701
14"-15" LEFT	CP103691	CP103698
16"-17" RIGHT	CP103695	CP103702
16"-17" LEFT	CP103691	CP103698
18"-19" RIGHT	CP103696	CP103703
18"-19" LEFT	CP103691	CP103698
20" RIGHT	CP103697	CP103704
20" LEFT	CP103691	CP103698


TABLE THREE - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	2GX, Aero X ADJ. HEIGHT 11.5"-15.5"	2GX, Aero X ADJ. HEIGHT 16.5"-20.5"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556



BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

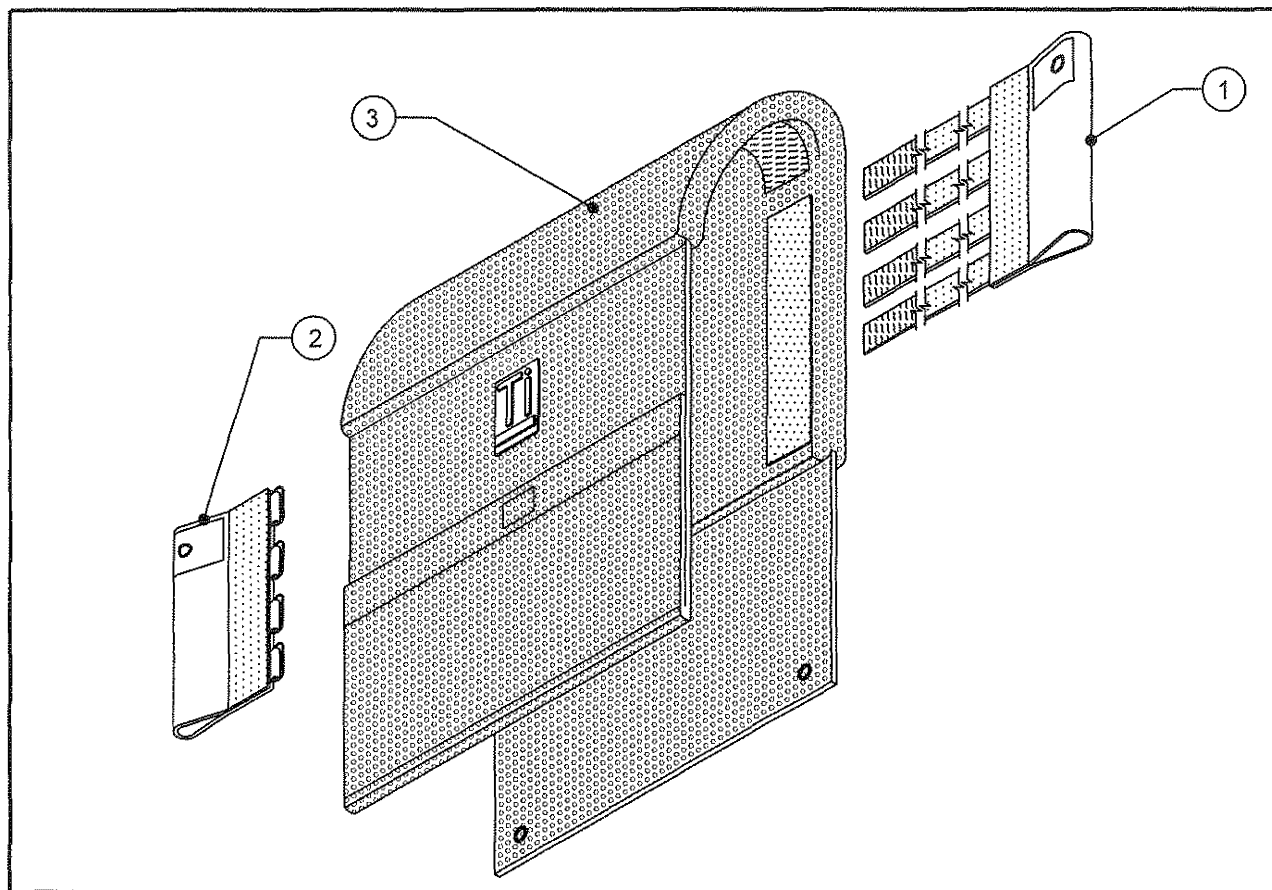

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE TWO	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE THREE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-3		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	2GX, Aero X ADJ. HEIGHT 11.5"-15.5"	2GX, Aero X ADJ. HEIGHT 16.5"-20.5"
12"	AS115755	AS115764
13"	AS115756	AS115765
14"	AS115757	AS115766
15"	AS115758	AS115767
16"	AS115759	AS115768
17"	AS115760	AS115769
18"	AS115761	AS115770
19"	AS115762	AS115771
20"	AS115763	AS115772

BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO - BACKREST EARS

CHAIR WIDTH	2GX, Aero X ADJ. HEIGHT 11.5"-15.5"	2GX, Aero X ADJ. HEIGHT 16.5"-20.5"
12"-13" RIGHT	CP103693	CP103700
12"-13" LEFT	CP103691	CP103698
14"-15" RIGHT	CP103694	CP103701
14"-15" LEFT	CP103691	CP103698
16"-17" RIGHT	CP103695	CP103702
16"-17" LEFT	CP103691	CP103698
18"-19" RIGHT	CP103696	CP103703
18"-19" LEFT	CP103691	CP103698
20" RIGHT	CP103697	CP103704
20" LEFT	CP103691	CP103698


TABLE THREE - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	2GX, Aero X ADJ. HEIGHT 11.5"-15.5"	2GX, Aero X ADJ. HEIGHT 16.5"-20.5"
12"	CP107023	CP107032
13"	CP107024	CP107033
14"	CP107025	CP107034
15"	CP107026	CP107035
16"	CP107027	CP107036
17"	CP107028	CP107037
18"	CP107029	CP107038
19"	CP107030	CP107039
20"	CP107031	CP107040



PADDED NYLON BACK UPHOLSTERY

TiLite™ X, SX, Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

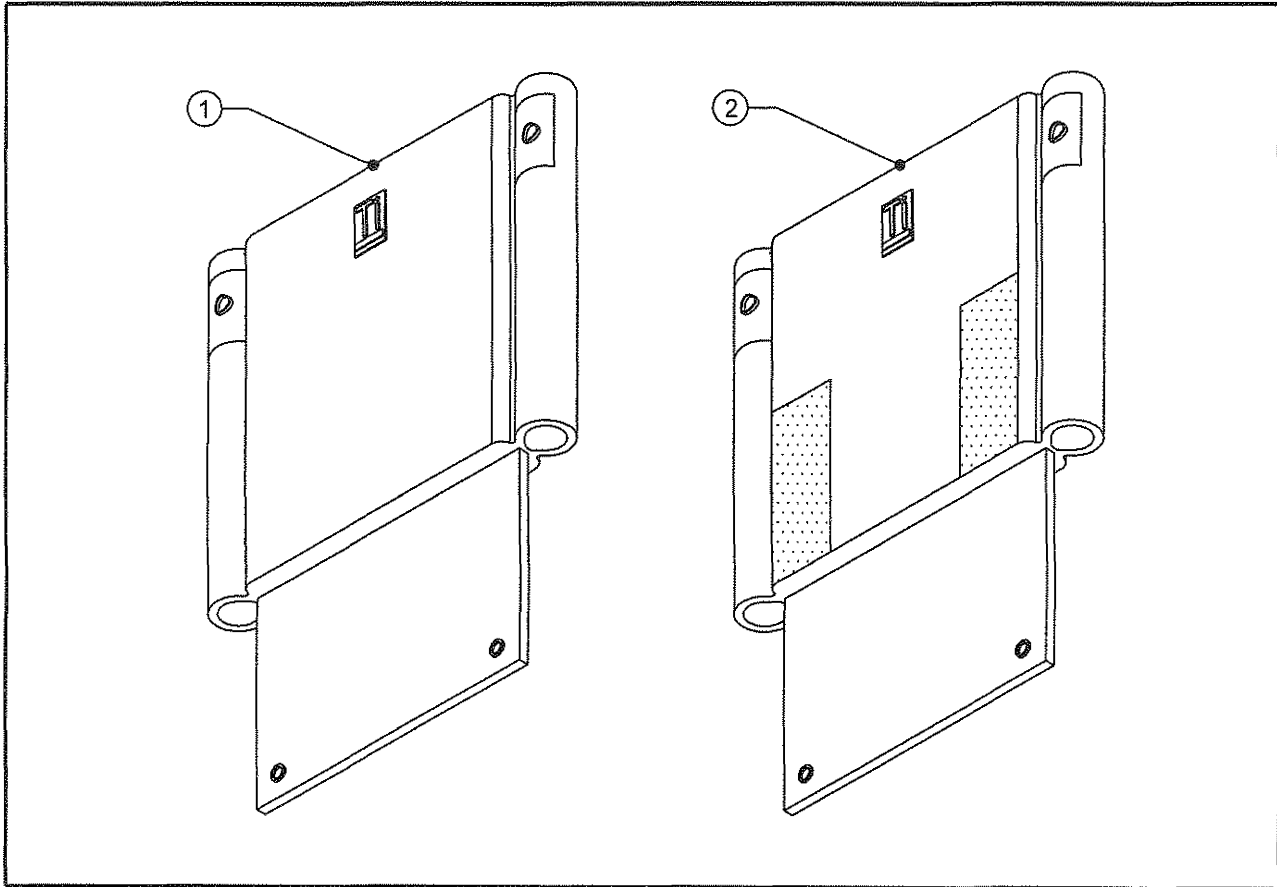


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	PADDED NYLON BACK PAD
2	1	TABLE ONE	PADDED NYLON BACK PAD W/CLOTH SIDE GUARDS

TABLE ONE

CHAIR WIDTH	STANDARD		W/ CLOTH SIDEGUARDS	
	ADJ. HEIGHT X, SX 11"-14" AERO X, 2GX 11"-15"	ADJ. HEIGHT X, SX 15"-18" AERO X, 2GX 16"-20"	ADJ. HEIGHT X, SX 11"-14" AERO X, 2GX 11"-15"	ADJ. HEIGHT X, SX 15"-18" AERO X, 2GX 16"-20"
12"	CP100272	CP100279	CP100779	CP100786
13"	CP100273	CP100280	CP100780	CP100787
14"	CP100274	CP100281	CP100781	CP100788
15"	CP100275	CP100282	CP100782	CP100789
16"	CP100276	CP100283	CP100783	CP100790
17"	CP100277	CP100284	CP100784	CP100791
18"	CP100278	CP100285	CP100785	CP100792
19"	CP101563	CP101565	CP101559	CP101561
20"	CP101564	CP101566	CP101560	CP101562

BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ XC, TX

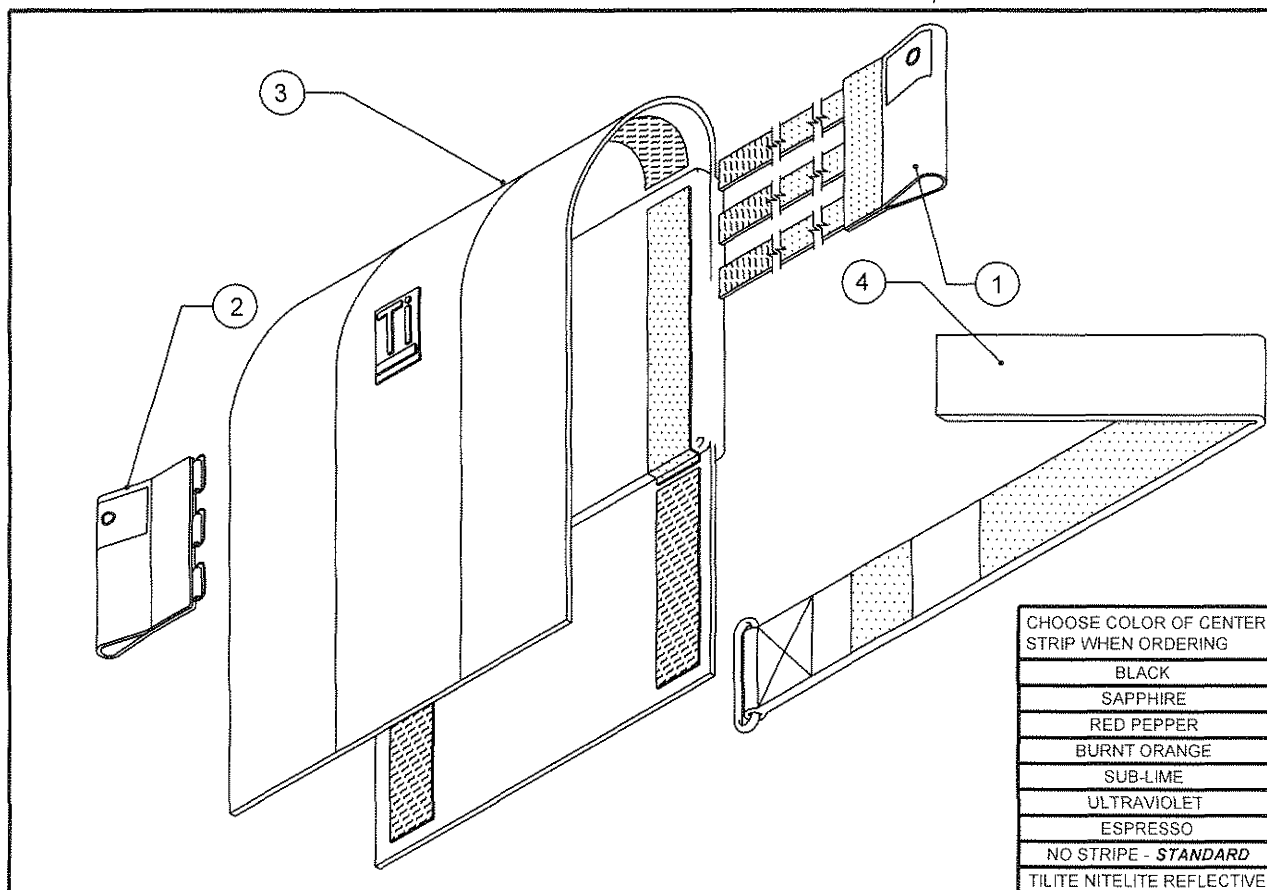

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	2-5	TABLE FIVE	BACK STRAP (SEE NEXT PAGE)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE

CHAIR WIDTH	STANDARD				PUSH HANDLES			
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16"	ADJ. HEIGHT 14"-18"	ADJ. HEIGHT 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16"	ADJ. HEIGHT 15"-18"	ADJ. HEIGHT 17"-20"
12"	AS101668	AS100532	AS100539	AS103097	AS101679	AS101821	AS101828	AS103115
13"	AS101669	AS100533	AS100540	AS103098	AS101680	AS101822	AS101829	AS103116
14"	AS101670	AS100534	AS100541	AS103099	AS101681	AS101823	AS101830	AS103117
15"	AS101671	AS100535	AS100542	AS103100	AS101682	AS101824	AS101831	AS103118
16"	AS101672	AS100536	AS100543	AS103101	AS101683	AS101825	AS101832	AS103119
17"	AS101673	AS100537	AS100544	AS103102	AS101684	AS101826	AS101833	AS103120
18"	AS101674	AS100538	AS100545	AS103103	AS101685	AS101827	AS101834	AS103121
19"	AS101675	AS101664	AS101666	AS103104	AS101686	AS101677	AS101690	AS103122
20"	AS101676	AS101665	AS101667	AS103105	AS101687	AS101678	AS101691	AS103123

BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ XC, TX (CONT.)

TABLE TWO

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16", 14"-18", 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16", 15"-18", 17"-20"
12"-13"	CP101502	CP100049	CP101496	CP100323
14"-15"	CP101504	CP100264	CP101497	CP100324
16"-17"	CP101505	CP100265	CP101498	CP100325
18"-19"	CP101506	CP100266	CP101499	CP100326
20"	CP101507	CP101508	CP101500	CP101494

TABLE THREE

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16", 14"-18", 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16", 15"-18", 17"-20"
ALL WIDTHS	CP101503	CP100050	CP101495	CP100322

TABLE FOUR

CHAIR WIDTH	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16"	ADJ. HEIGHT 14"-18"	ADJ. HEIGHT 16"-20"
12"	CP101212	CP100833	CP100897	CP102893
13"	CP101213	CP100834	CP100868	CP102894
14"	CP101214	CP100835	CP100869	CP102895
15"	CP101215	CP100836	CP100870	CP102896
16"	CP101216	CP100837	CP100871	CP102897
17"	CP101217	CP100838	CP100872	CP102898
18"	CP101218	CP100839	CP100873	CP102899
19"	CP101570	CP101595	CP101567	CP102900
20"	CP101571	CP101596	CP101568	CP102901

TABLE FIVE

CHAIR WIDTH	
12"-13"	CP100492
14"-15"	CP100494
16"-17"	CP100496
18"-19"	CP100498
20"	CP101522

*NOTE FOR ITEM 4:

8"-12" BACK HEIGHTS USE TWO RIGID BACK STRAPS


12"-16" BACK HEIGHTS USE THREE RIGID BACK STRAPS

14"-18" BACK HEIGHTS USE FOUR RIGID BACK STRAPS

16"-20" BACK HEIGHTS USE FIVE RIGID BACK STRAPS

BREATHABLE BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ XC, TX

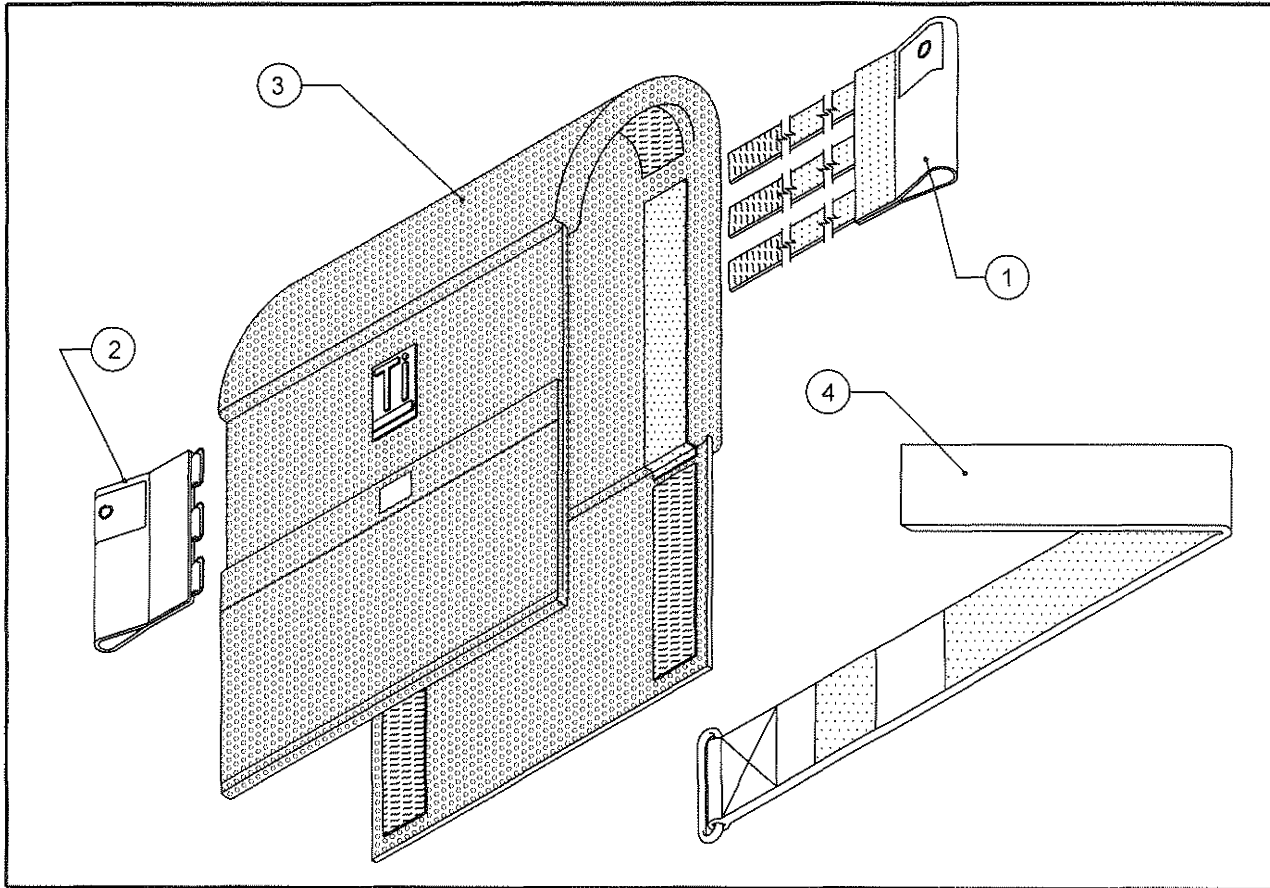

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	2-5	TABLE FIVE	BACK STRAP (SEE NEXT PAGE)
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY W/ PUSH ASSEMBLY
1-4**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE

CHAIR WIDTH	STANDARD				PUSH HANDLES			
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16"	ADJ. HEIGHT 14"-18"	ADJ. HEIGHT 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16"	ADJ. HEIGHT 15"-18"	ADJ. HEIGHT 17"-20"
12"	AS115476	AS115485	AS115494	AS115503	AS115512	AS115521	AS115530	AS115539
13"	AS115477	AS115486	AS115495	AS115504	AS115513	AS115522	AS115531	AS115540
14"	AS115478	AS115487	AS115496	AS115505	AS115514	AS115523	AS115532	AS115541
15"	AS115479	AS115488	AS115497	AS115506	AS115515	AS115524	AS115533	AS115542
16"	AS115480	AS115489	AS115498	AS115507	AS115516	AS115525	AS115534	AS115543
17"	AS115481	AS115490	AS115499	AS115508	AS115517	AS115526	AS115535	AS115544
18"	AS115482	AS115491	AS115500	AS115509	AS115518	AS115527	AS115536	AS115545
19"	AS115483	AS115492	AS115501	AS115510	AS115519	AS115528	AS115537	AS115546
20"	AS115484	AS115493	AS115502	AS115511	AS115520	AS115529	AS115538	AS115547



BREATHABLE BACK UPHOLSTERY

TENSION ADJUSTABLE FOR TiLite™ XC, TX (CONT.)

TABLE TWO

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16", 14"-18", 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16", 15"-18", 17"-20"
12"-13"	CP101502	CP100049	CP101496	CP100323
14"-15"	CP101504	CP100264	CP101497	CP100324
16"-17"	CP101505	CP100265	CP101498	CP100325
18"-19"	CP101506	CP100266	CP101499	CP100326
20"	CP101507	CP101508	CP101500	CP101494

TABLE THREE

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16", 14"-18", 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16", 15"-18", 17"-20"
ALL WIDTHS	CP101503	CP100050	CP101495	CP100322

TABLE FOUR

CHAIR WIDTH	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16"	ADJ. HEIGHT 14"-18"	ADJ. HEIGHT 16"-20"
12"	CP107041	CP107050	CP107059	CP107068
13"	CP107042	CP107051	CP107060	CP107069
14"	CP107043	CP107052	CP107061	CP107070
15"	CP107044	CP107053	CP107062	CP107071
16"	CP107045	CP107054	CP107063	CP107072
17"	CP107046	CP107055	CP107064	CP107073
18"	CP107047	CP107056	CP107065	CP107074
19"	CP107048	CP107057	CP107066	CP107075
20"	CP107049	CP107058	CP107067	CP107076

TABLE FIVE

CHAIR WIDTH	
12"-13"	CP100492
14"-15"	CP100494
16"-17"	CP100496
18"-19"	CP100498
20"	CP101522

*NOTE FOR ITEM 4:

8"-12" BACK HEIGHTS USE TWO RIGID BACK STRAPS

12"-16" BACK HEIGHTS USE THREE RIGID BACK STRAPS


14"-18" BACK HEIGHTS USE FOUR RIGID BACK STRAPS

16"-20" BACK HEIGHTS USE FIVE RIGID BACK STRAPS



TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES TiLite™ TX

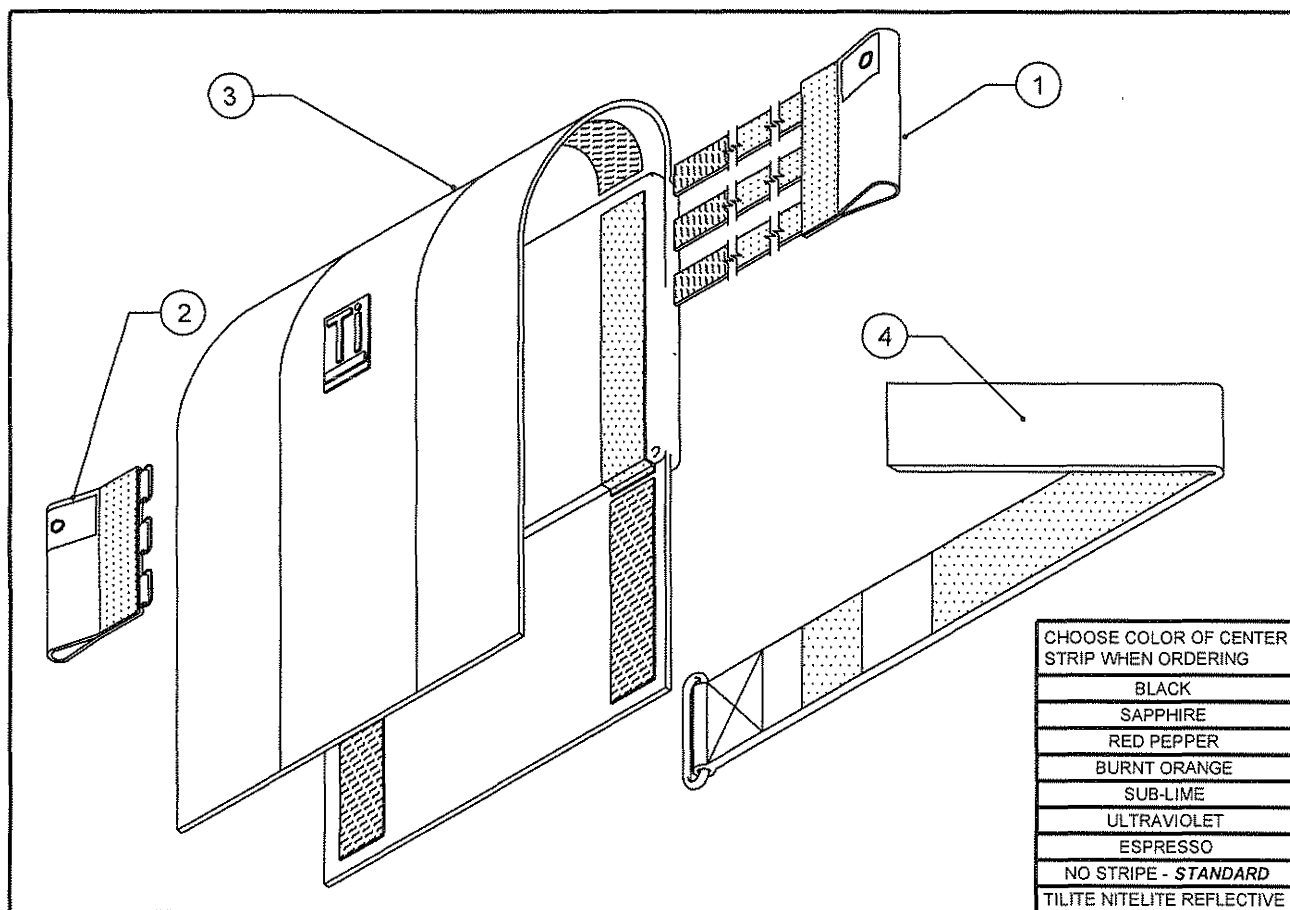

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 16"-20.5"
12"	AS104067	AS104076
13"	AS104068	AS104077
14"	AS104069	AS104078
15"	AS104070	AS104079
16"	AS104071	AS104080
17"	AS104072	AS104081
18"	AS104073	AS104082
19"	AS104074	AS104083
20"	AS104075	AS104084

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4
14"-18.5" BACK HEIGHTS USE TWO 2" BACK STRAPS
16"-20.5" BACK HEIGHTS USE THREE 2" BACK STRAPS

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES TiLite™ TX

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 16"-20.5"
12"-13" RIGHT	CP103717	CP103717
12"-13" LEFT	CP103715	CP103715
14"-15" RIGHT	CP103718	CP103718
14"-15" LEFT	CP103715	CP103715
16"-17" RIGHT	CP103719	CP103719
16"-17" LEFT	CP103715	CP103715
18"-19" RIGHT	CP103720	CP103720
18"-19" LEFT	CP103715	CP103715
20" RIGHT	CP103721	CP103721
20" LEFT	CP103715	CP103715


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	14"-18.5" SEAT BACK HEIGHT	16"-20.5" SEAT BACK HEIGHT
12"	CP100897	CP102893
13"	CP100868	CP102894
14"	CP100869	CP102895
15"	CP100870	CP102896
16"	CP100871	CP102897
17"	CP100872	CP102898
18"	CP100873	CP102899
19"	CP101567	CP102900
20"	CP101568	CP102901



BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES TiLite™ TX

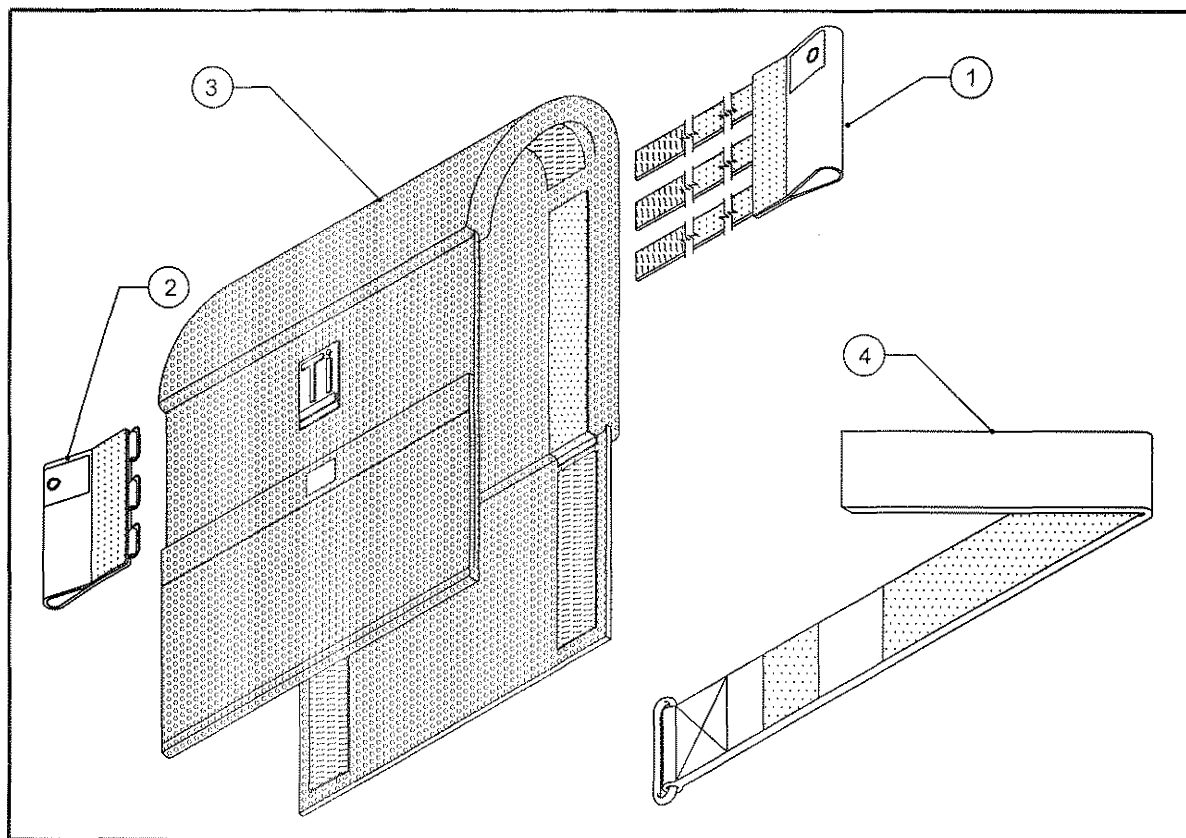

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4*	1-3	TABLE TWO	BACK STRAP
1-4		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 16"-20.5"
12"	AS115710	AS115719
13"	AS115711	AS115720
14"	AS115712	AS115721
15"	AS115713	AS115722
16"	AS115714	AS115723
17"	AS115715	AS115724
18"	AS115716	AS115725
19"	AS115717	AS115726
20"	AS115718	AS115727

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 4

14"-18.5" BACK HEIGHTS USE TWO 2" BACK STRAPS

16"-20.5" BACK HEIGHTS USE THREE 2" BACK STRAPS

BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES TiLite™ TX

TABLE THREE - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT 14"-18.5"	ADJ. HEIGHT 16"-20.5"
12"-13" RIGHT	CP103717	CP103717
12"-13" LEFT	CP103715	CP103715
14"-15" RIGHT	CP103718	CP103718
14"-15" LEFT	CP103715	CP103715
16"-17" RIGHT	CP103719	CP103719
16"-17" LEFT	CP103715	CP103715
18"-19" RIGHT	CP103720	CP103720
18"-19" LEFT	CP103715	CP103715
20" RIGHT	CP103721	CP103721
20" LEFT	CP103715	CP103715


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	14"-18.5" SEAT BACK HEIGHT	16"-20.5" SEAT BACK HEIGHT
12"	CP107059	CP107068
13"	CP107060	CP107069
14"	CP107061	CP107070
15"	CP107062	CP107071
16"	CP107063	CP107072
17"	CP107064	CP107073
18"	CP107065	CP107074
19"	CP107066	CP107075
20"	CP107067	CP107076



BACK UPHOLSTERY

VELCRO ADJUSTABLE FOR TiLite™ XC, TX

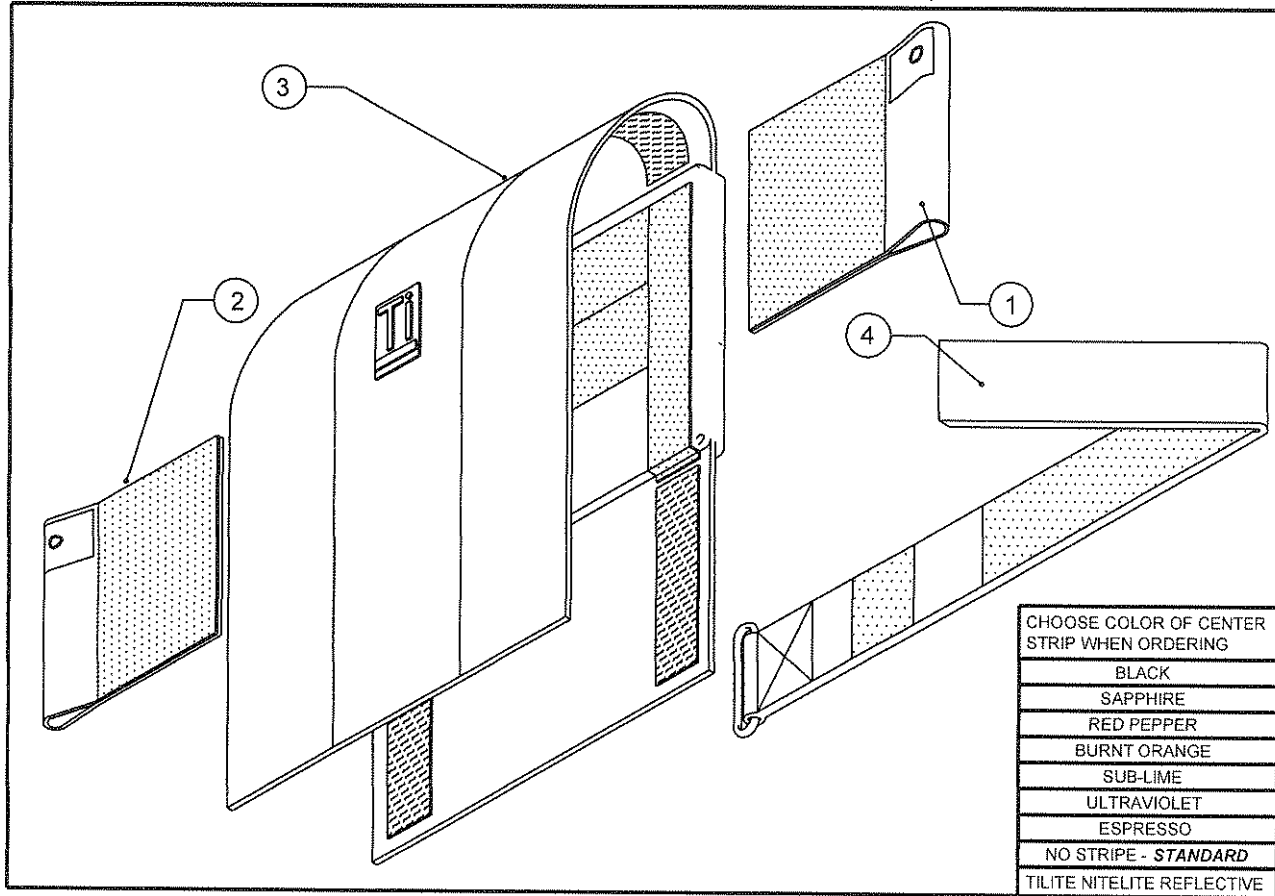

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE TWO	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4	2	TABLE FIVE	BACK STRAP (SEE NEXT PAGE)
1-4		TABLE ONE	VELCRO ADJ, BACK UPHOLSTERY ASSEMBLY

TABLE ONE

CHAIR WIDTH	STANDARD				PUSH HANDLES			
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16"	ADJ. HEIGHT 14"-18"	ADJ. HEIGHT 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16"	ADJ. HEIGHT 15"-18"	ADJ. HEIGHT 17"-20"
12"	AS101640	AS100518	AS100525	AS103106	AS101653	AS100305	AS100312	AS103124
13"	AS101641	AS100519	AS100526	AS103107	AS101654	AS100306	AS100313	AS103125
14"	AS101642	AS100520	AS100527	AS103108	AS101655	AS100307	AS100314	AS103126
15"	AS101643	AS100521	AS100528	AS103109	AS101656	AS100308	AS100315	AS103127
16"	AS101644	AS100522	AS100529	AS103110	AS101657	AS100309	AS100316	AS103128
17"	AS101645	AS100523	AS100530	AS103111	AS101658	AS100310	AS100317	AS103129
18"	AS101646	AS100524	AS100531	AS103112	AS101659	AS100311	AS100318	AS103130
19"	AS101647	AS101638	AS101649	AS103113	AS101660	AS101651	AS101662	AS103131
20"	AS101648	AS101639	AS101650	AS103114	AS101661	AS101652	AS101663	AS103132

NOTE:

8"-12" BACK HEIGHTS USE TWO RIGID BACK STRAPS
 12"-16" BACK HEIGHTS USE THREE RIGID BACK STRAPS
 14"-18" BACK HEIGHTS USE FOUR RIGID BACK STRAPS
 16"-20" BACK HEIGHTS USE FIVE RIGID BACK STRAPS

BACK UPHOLSTERY

VELCRO ADJUSTABLE FOR TiLite™ XC, TX (CONT.)

TABLE TWO

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16", 14"-18", 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16", 15"-18", 17"-20"
12"-13"	CP101510	CP100450	CP101484	CP100508
14"-18"	CP101511	CP100451	CP101486	CP100509
19"-20"	CP101524	CP101526	CP101530	CP101528

TABLE THREE

CHAIR WIDTH	STANDARD		PUSH HANDLES	
	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16", 14"-18", 16"-20"	ADJ. HEIGHT 9"-12"	ADJ. HEIGHT 13"-16", 15"-18", 17"-20"
12"-13"	CP101509	CP100514	CP101485	CP100673
14"-18"	CP101512	CP100454	CP101487	CP100507
19"-20"	CP101523	CP101525	CP101529	CP101527

TABLE FOUR

CHAIR WIDTH	ADJ. HEIGHT 8"-12"	ADJ. HEIGHT 12"-16"	ADJ. HEIGHT 14"-18"	ADJ. HEIGHT 16"-20"
12"	CP101572	CP100840	CP100874	CP102902
13"	CP101573	CP100841	CP100875	CP102903
14"	CP101574	CP100842	CP100876	CP102904
15"	CP101575	CP100843	CP100877	CP102905
16"	CP101576	CP100844	CP100878	CP102906
17"	CP101577	CP100845	CP100879	CP102907
18"	CP101578	CP100846	CP100880	CP102908
19"	CP101579	CP101592	CP101600	CP102909
20"	CP101580	CP101593	CP101601	CP102910

TABLE FIVE

CHAIR WIDTH	
12"-13"	CP100492
14"-15"	CP100494
16"-17"	CP100496
18"-19"	CP100498
20"	CP101522


NOTE:

8"-12" BACK HEIGHTS USE TWO RIGID BACK STRAPS
 12"-16" BACK HEIGHTS USE THREE RIGID BACK STRAPS
 14"-18" BACK HEIGHTS USE FOUR RIGID BACK STRAPS
 16"-20" BACK HEIGHTS USE FIVE RIGID BACK STRAPS

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR

TiLite™ X, SX

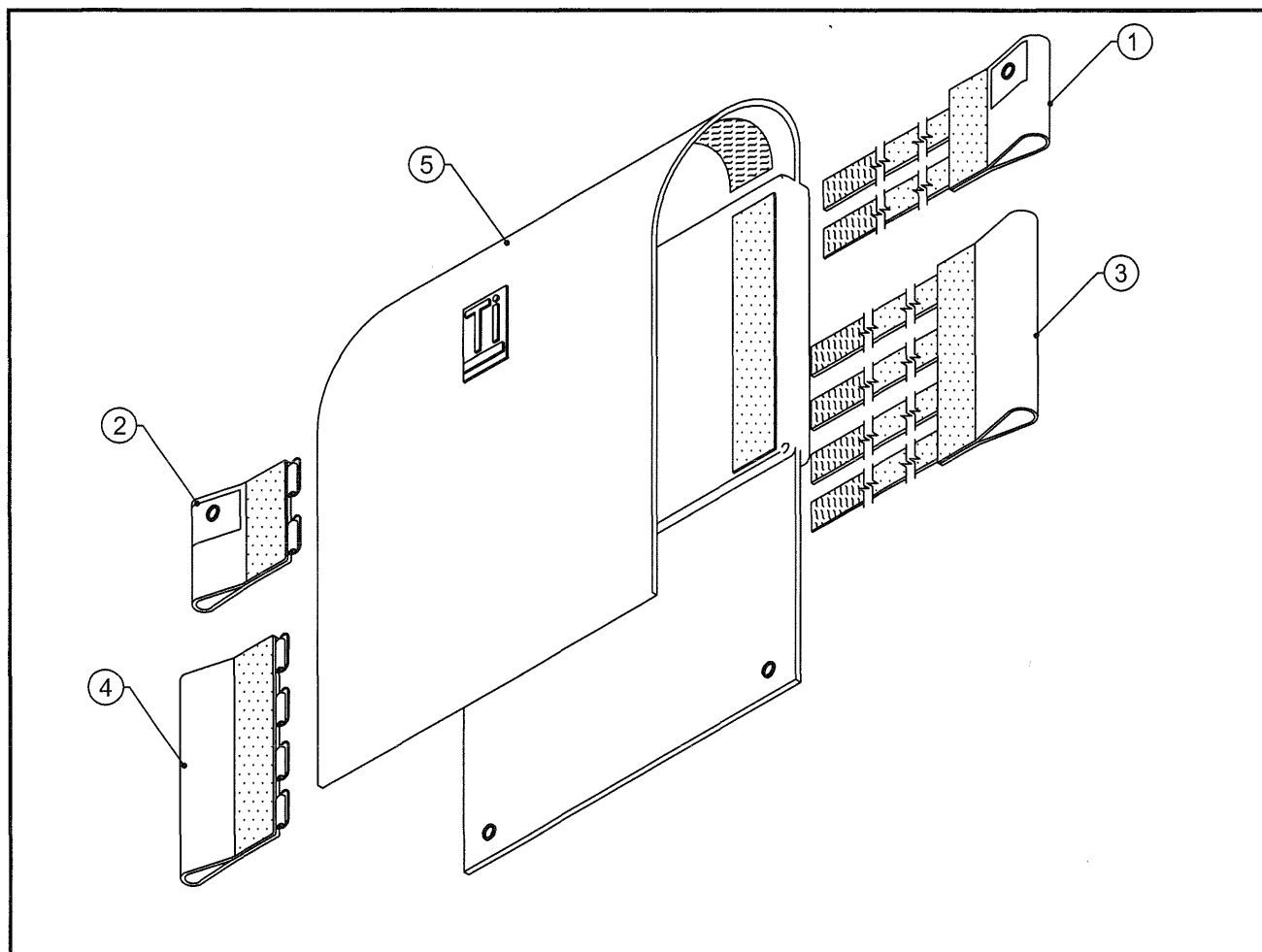

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP101503	BACKREST EAR, LEFT
3	1	TABLE TWO	BACKREST LOWER EAR, RIGHT (SEE NEXT PAGE)
4	1	TABLE THREE	BACKREST LOWER EAR, LEFT (SEE NEXT PAGE)
5	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-5		TABLE FIVE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ X, SX

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP101502
14"-15"	CP101504
16"-17"	CP101505
18"-19"	CP101506
20"	CP101507

TABLE TWO - BACKREST LOWER EAR, RIGHT

SEAT WIDTH	BACK HEIGHT 11"-14"	BACK HEIGHT 15"-18"
12"-13"	CP103265	CP103270
14"-15"	CP103266	CP103271
16"-17"	CP103267	CP103272
18"-19"	CP103268	CP103273
20"	CP103269	CP103274

TABLE THREE - BACKREST LOWER EAR, LEFT

SEAT WIDTH	BACK HEIGHT 11"-14"	BACK HEIGHT 15"-18"
12"-13"	CP103275	CP103276
14"-15"	CP103275	CP103276
16"-17"	CP103275	CP103276
18"-19"	CP103275	CP103276
20"	CP103275	CP103276

TABLE FOUR - BACK PAD TENSION ADJUSTABLE


SEAT WIDTH	BACK HEIGHT 11"-14"	BACK HEIGHT 15"-18"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556

TABLE FIVE - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	SHORT 11"-14"	MEDIUM 15"-18"
12"	AS103486	AS103495
13"	AS103487	AS103496
14"	AS103488	AS103497
15"	AS103489	AS103498
16"	AS103490	AS103499
17"	AS103491	AS103500
18"	AS103492	AS103501
19"	AS103493	AS103502
20"	AS103494	AS103503

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

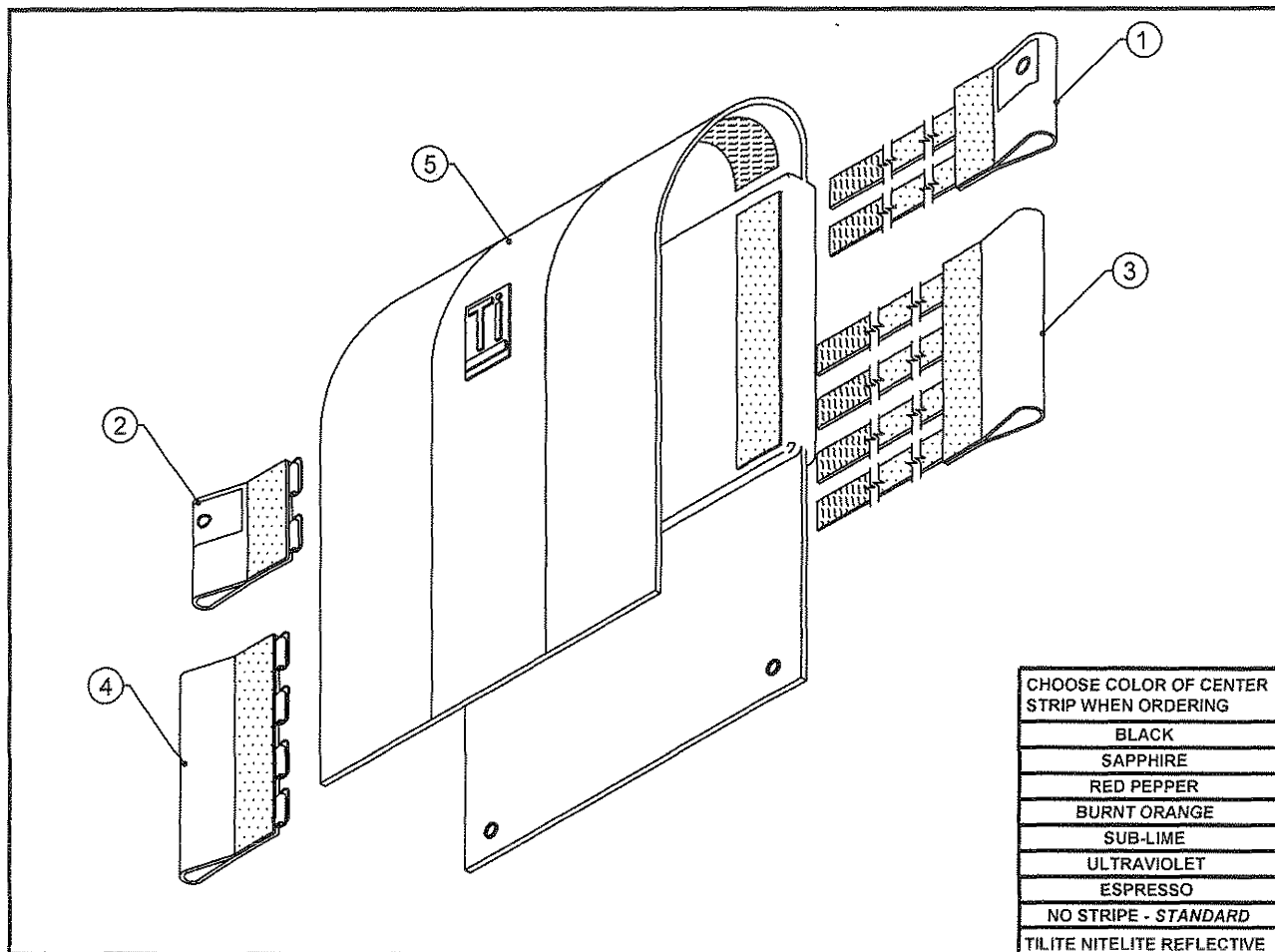

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP101503	BACKREST EAR, LEFT
3	1	TABLE TWO	BACKREST LOWER EAR, RIGHT (SEE NEXT PAGE)
4	1	TABLE THREE	BACKREST LOWER EAR, LEFT (SEE NEXT PAGE)
5	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-5		TABLE FIVE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP101502
14"-15"	CP101504
16"-17"	CP101505
18"-19"	CP101506
20"	CP101507

TABLE TWO - BACKREST LOWER EAR, RIGHT

SEAT WIDTH	BACK HEIGHT 11"-15"	BACK HEIGHT 16"-20"
12"-13"	CP103265	CP103270
14"-15"	CP103266	CP103271
16"-17"	CP103267	CP103272
18"-19"	CP103268	CP103273
20"	CP103269	CP103274

TABLE THREE - BACKREST LOWER EAR, LEFT

SEAT WIDTH	BACK HEIGHT 11"-15"	BACK HEIGHT 16"-20"
12"-13"	CP103275	CP103276
14"-15"	CP103275	CP103276
16"-17"	CP103275	CP103276
18"-19"	CP103275	CP103276
20"	CP103275	CP103276

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	BACK HEIGHT 11"-15"	BACK HEIGHT 16"-20"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556

TABLE FIVE - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES


SEAT WIDTH	SHORT 11"-15"	MEDIUM 16"-20"
12"	AS103486	AS103495
13"	AS103487	AS103496
14"	AS103488	AS103497
15"	AS103489	AS103498
16"	AS103490	AS103499
17"	AS103491	AS103500
18"	AS103492	AS103501
19"	AS103493	AS103502
20"	AS103494	AS103503



Parts Manual

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ X, SX

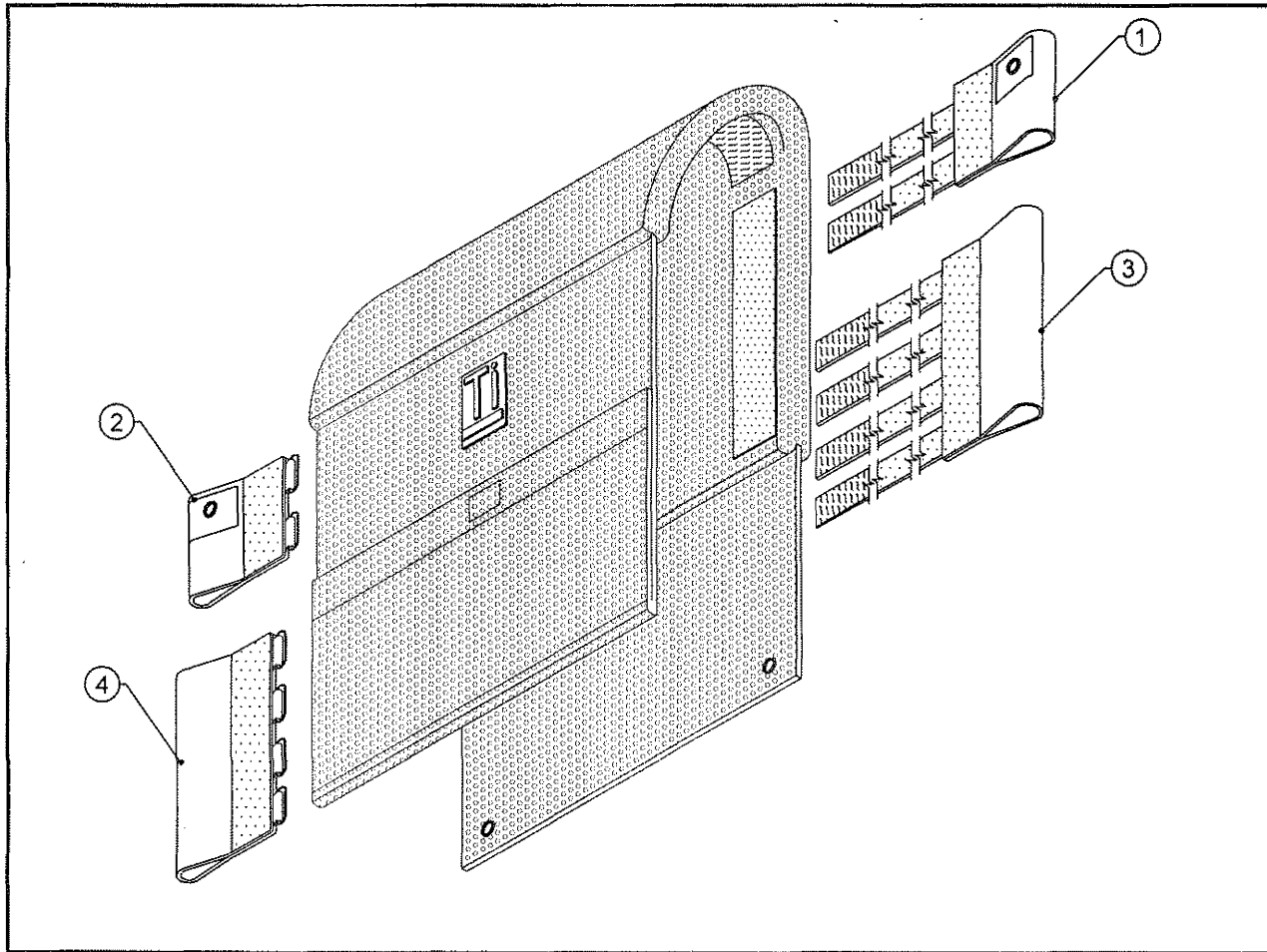

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP101503	BACKREST EAR, LEFT
3	1	TABLE TWO	BACKREST LOWER EAR, RIGHT (SEE NEXT PAGE)
4	1	TABLE THREE	BACKREST LOWER EAR, LEFT (SEE NEXT PAGE)
5	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-5		TABLE FIVE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE



Parts Manual

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ X, SX

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP101502
14"-15"	CP101504
16"-17"	CP101505
18"-19"	CP101506
20"	CP101507

TABLE TWO - BACKREST LOWER EAR, RIGHT

SEAT WIDTH	BACK HEIGHT 11"-14"	BACK HEIGHT 15"-18"
12"-13"	CP103265	CP103270
14"-15"	CP103266	CP103271
16"-17"	CP103267	CP103272
18"-19"	CP103268	CP103273
20"	CP103269	CP103274

TABLE THREE - BACKREST LOWER EAR, LEFT

SEAT WIDTH	BACK HEIGHT 11"-14"	BACK HEIGHT 15"-18"
12"-13"	CP103275	CP103276
14"-15"	CP103275	CP103276
16"-17"	CP103275	CP103276
18"-19"	CP103275	CP103276
20"	CP103275	CP103276

TABLE FOUR - BACK PAD TENSION ADJUSTABLE


SEAT WIDTH	BACK HEIGHT 11"-14"	BACK HEIGHT 15"-18"
12"	CP107023	CP107032
13"	CP107024	CP107033
14"	CP107025	CP107034
15"	CP107026	CP107035
16"	CP107027	CP107036
17"	CP107028	CP107037
18"	CP107029	CP107038
19"	CP107030	CP107039
20"	CP107031	CP107040

TABLE FIVE - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	SHORT 11"-14"	MEDIUM 15"-18"
12"	AS115548	AS115557
13"	AS115549	AS115558
14"	AS115550	AS115559
15"	AS115551	AS115560
16"	AS115552	AS115561
17"	AS115553	AS115562
18"	AS115554	AS115563
19"	AS115555	AS115564
20"	AS115556	AS115565

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

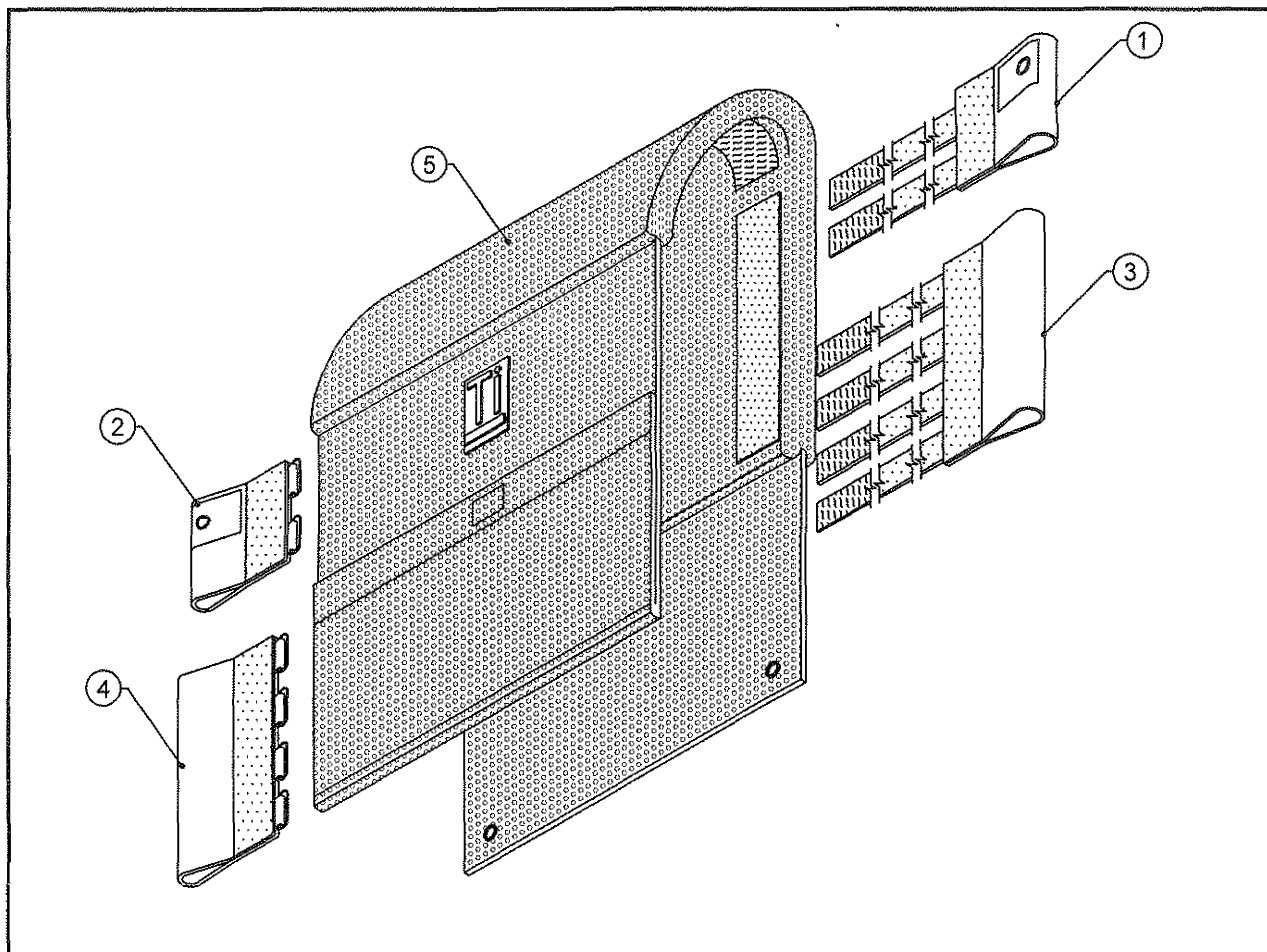

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP101503	BACKREST EAR, LEFT
3	1	TABLE TWO	BACKREST LOWER EAR, RIGHT (SEE NEXT PAGE)
4	1	TABLE THREE	BACKREST LOWER EAR, LEFT (SEE NEXT PAGE)
5	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-5		TABLE FIVE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP101502
14"-15"	CP101504
16"-17"	CP101505
18"-19"	CP101506
20"	CP101507

TABLE TWO - BACKREST LOWER EAR, RIGHT

SEAT WIDTH	BACK HEIGHT 11"-15"	BACK HEIGHT 16"-20"
12"-13"	CP103265	CP103270
14"-15"	CP103266	CP103271
16"-17"	CP103267	CP103272
18"-19"	CP103268	CP103273
20"	CP103269	CP103274

TABLE THREE - BACKREST LOWER EAR, LEFT

SEAT WIDTH	BACK HEIGHT 11"-15"	BACK HEIGHT 16"-20"
12"-13"	CP103275	CP103276
14"-15"	CP103275	CP103276
16"-17"	CP103275	CP103276
18"-19"	CP103275	CP103276
20"	CP103275	CP103276

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	BACK HEIGHT 11"-15"	BACK HEIGHT 16"-20"
12"	CP107023	CP107032
13"	CP107024	CP107033
14"	CP107025	CP107034
15"	CP107026	CP107035
16"	CP107027	CP107036
17"	CP107028	CP107037
18"	CP107029	CP107038
19"	CP107030	CP107039
20"	CP107031	CP107040


TABLE FIVE - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	SHORT 11"-15"	MEDIUM 16"-20"
12"	AS115548	AS115557
13"	AS115549	AS115558
14"	AS115550	AS115559
15"	AS115551	AS115560
16"	AS115552	AS115561
17"	AS115553	AS115562
18"	AS115554	AS115563
19"	AS115555	AS115564
20"	AS115556	AS115565

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR

TiLite™ X, SX

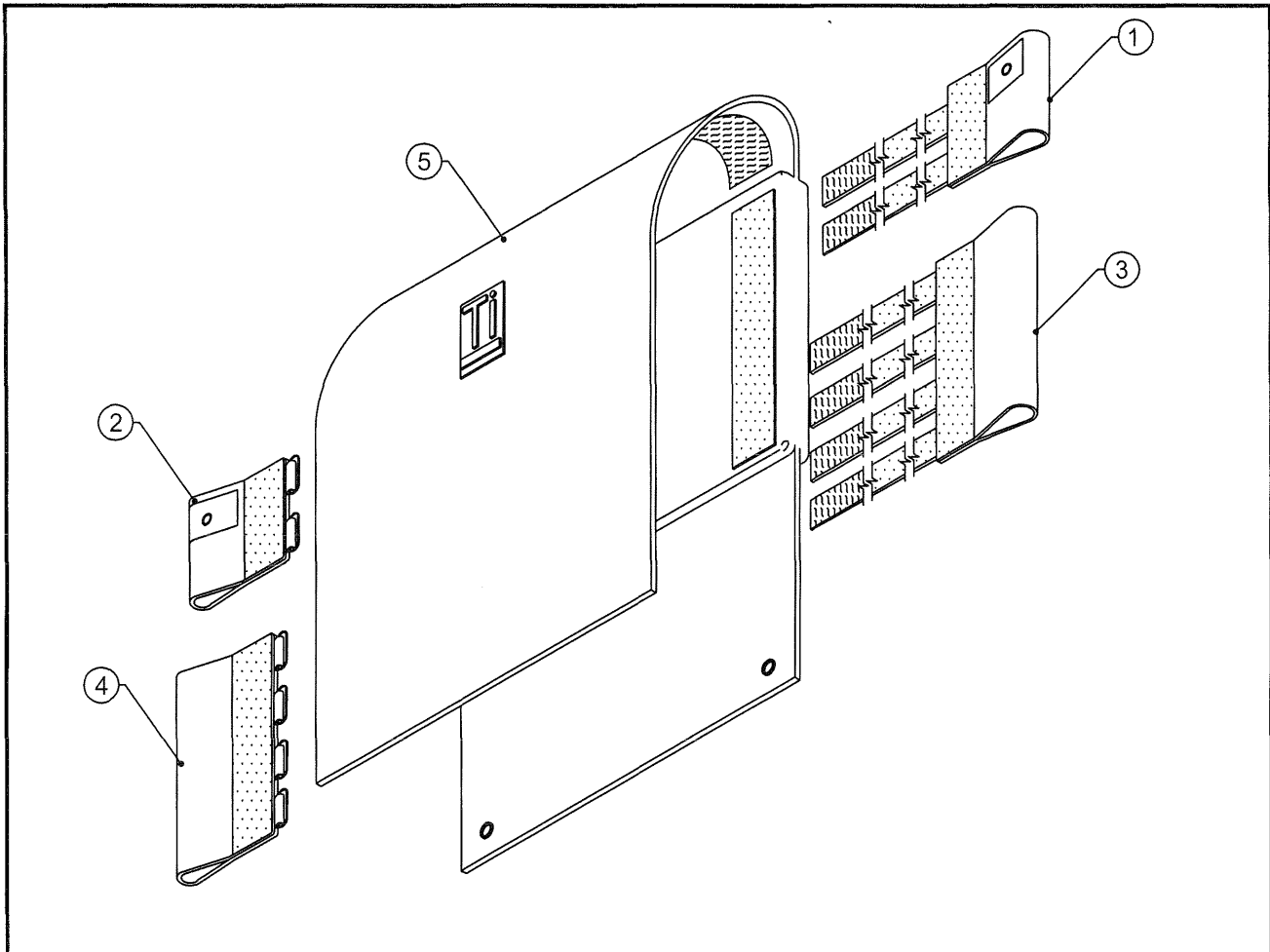

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP103705	BACKREST EAR, LEFT
3	1	TABLE TWO	BACKREST LOWER EAR, RIGHT (SEE NEXT PAGE)
4	1	TABLE THREE	BACKREST LOWER EAR, LEFT (SEE NEXT PAGE)
5	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-5		TABLE FIVE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR

TiLite™ X, SX

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP103707
14"-15"	CP103708
16"-17"	CP103709
18"-19"	CP103710
20"	CP103711

TABLE TWO - BACKREST LOWER EAR, RIGHT

SEAT WIDTH	X, SX BACK HEIGHT 11.5"-14.5"	X, SX BACK HEIGHT 15.5"-18.5"
12"-13"	CP103265	CP103270
14"-15"	CP103266	CP103271
16"-17"	CP103267	CP103272
18"-19"	CP103268	CP103273
20"	CP103269	CP103274

TABLE THREE - BACKREST LOWER EAR, LEFT

SEAT WIDTH	X, SX BACK HEIGHT 11.5"-14.5"	X, SX BACK HEIGHT 15.5"-18.5"
12"-13"	CP103275	CP103276
14"-15"	CP103275	CP103276
16"-17"	CP103275	CP103276
18"-19"	CP103275	CP103276
20"	CP103275	CP103276

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	X, SX BACK HEIGHT 11.5"-14.5"	X, SX BACK HEIGHT 15.5"-18.5"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556


TABLE FIVE - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	X, SX BACK HEIGHT 11.5"-14.5"	X, SX BACK HEIGHT 15.5"-18.5"
12"	AS104103	AS104112
13"	AS104104	AS104113
14"	AS104105	AS104114
15"	AS104106	AS104115
16"	AS104107	AS104116
17"	AS104108	AS104117
18"	AS104109	AS104118
19"	AS104110	AS104119
20"	AS104111	AS104120

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR

TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

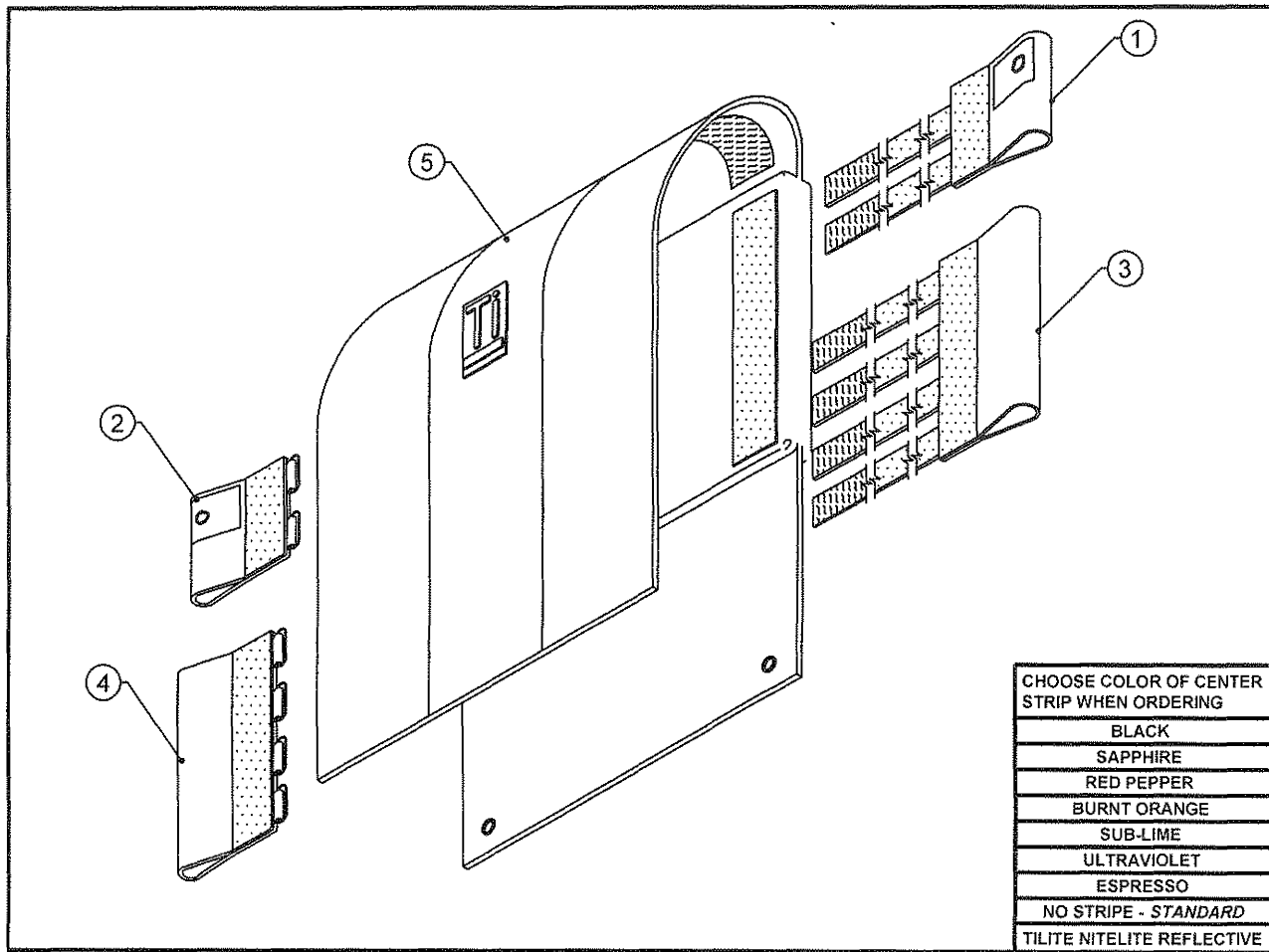

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP103705	BACKREST EAR, LEFT
3	1	TABLE TWO	BACKREST LOWER EAR, RIGHT (SEE NEXT PAGE)
4	1	TABLE THREE	BACKREST LOWER EAR, LEFT (SEE NEXT PAGE)
5	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-5		TABLE FIVE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP103707
14"-15"	CP103708
16"-17"	CP103709
18"-19"	CP103710
20"	CP103711

TABLE TWO - BACKREST LOWER EAR, RIGHT

SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"-13"	CP103265	CP103270
14"-15"	CP103266	CP103271
16"-17"	CP103267	CP103272
18"-19"	CP103268	CP103273
20"	CP103269	CP103274

TABLE THREE - BACKREST LOWER EAR, LEFT

SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"-13"	CP103275	CP103276
14"-15"	CP103275	CP103276
16"-17"	CP103275	CP103276
18"-19"	CP103275	CP103276
20"	CP103275	CP103276

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"	CP100799	CP100584
13"	CP100800	CP100585
14"	CP100801	CP100586
15"	CP100802	CP100587
16"	CP100803	CP100588
17"	CP100804	CP100589
18"	CP100805	CP100590
19"	CP101553	CP101555
20"	CP101554	CP101556


TABLE FIVE - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"	AS104103	AS104112
13"	AS104104	AS104113
14"	AS104105	AS104114
15"	AS104106	AS104115
16"	AS104107	AS104116
17"	AS104108	AS104117
18"	AS104109	AS104118
19"	AS104110	AS104119
20"	AS104111	AS104120



BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

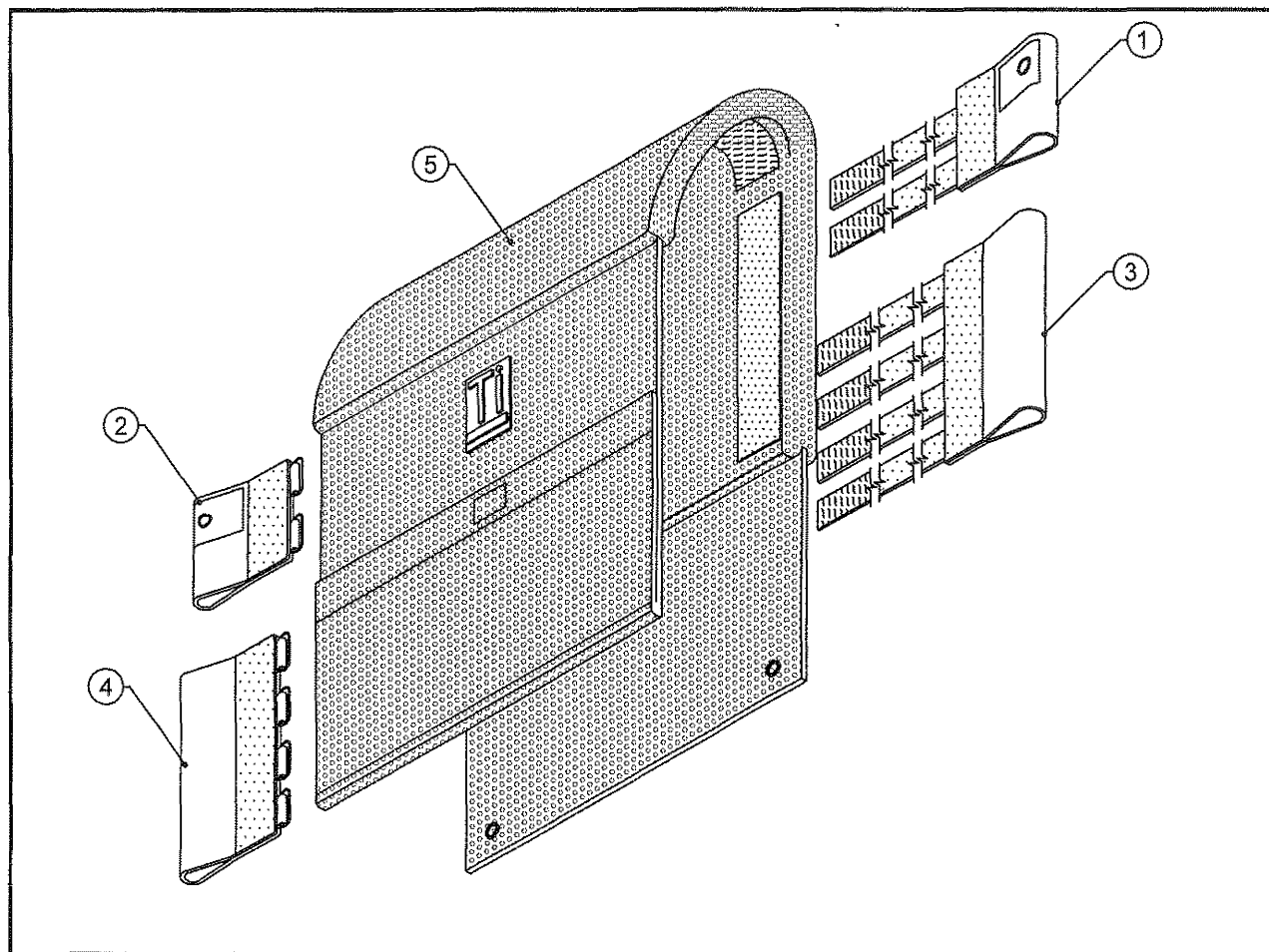

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP103705	BACKREST EAR, LEFT
3	1	TABLE TWO	BACKREST LOWER EAR, RIGHT (SEE NEXT PAGE)
4	1	TABLE THREE	BACKREST LOWER EAR, LEFT (SEE NEXT PAGE)
5	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-5		TABLE FIVE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR
TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP103707
14"-15"	CP103708
16"-17"	CP103709
18"-19"	CP103710
20"	CP103711

TABLE TWO - BACKREST LOWER EAR, RIGHT

SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"-13"	CP103265	CP103270
14"-15"	CP103266	CP103271
16"-17"	CP103267	CP103272
18"-19"	CP103268	CP103273
20"	CP103269	CP103274

TABLE THREE - BACKREST LOWER EAR, LEFT

SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"-13"	CP103275	CP103276
14"-15"	CP103275	CP103276
16"-17"	CP103275	CP103276
18"-19"	CP103275	CP103276
20"	CP103275	CP103276

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"	CP107023	CP107032
13"	CP107024	CP107033
14"	CP107025	CP107034
15"	CP107026	CP107035
16"	CP107027	CP107036
17"	CP107028	CP107037
18"	CP107029	CP107038
19"	CP107030	CP107039
20"	CP107031	CP107040

TABLE FIVE - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES


SEAT WIDTH	2GX, Aero X BACK HEIGHT 11.5"-15.5"	2GX, Aero X BACK HEIGHT 16.5"-20.5"
12"	AS115773	AS115782
13"	AS115774	AS115783
14"	AS115775	AS115784
15"	AS115776	AS115785
16"	AS115777	AS115786
17"	AS115778	AS115787
18"	AS115779	AS115788
19"	AS115780	AS115789
20"	AS115781	AS115790



TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR

TiLite™ XC, TX

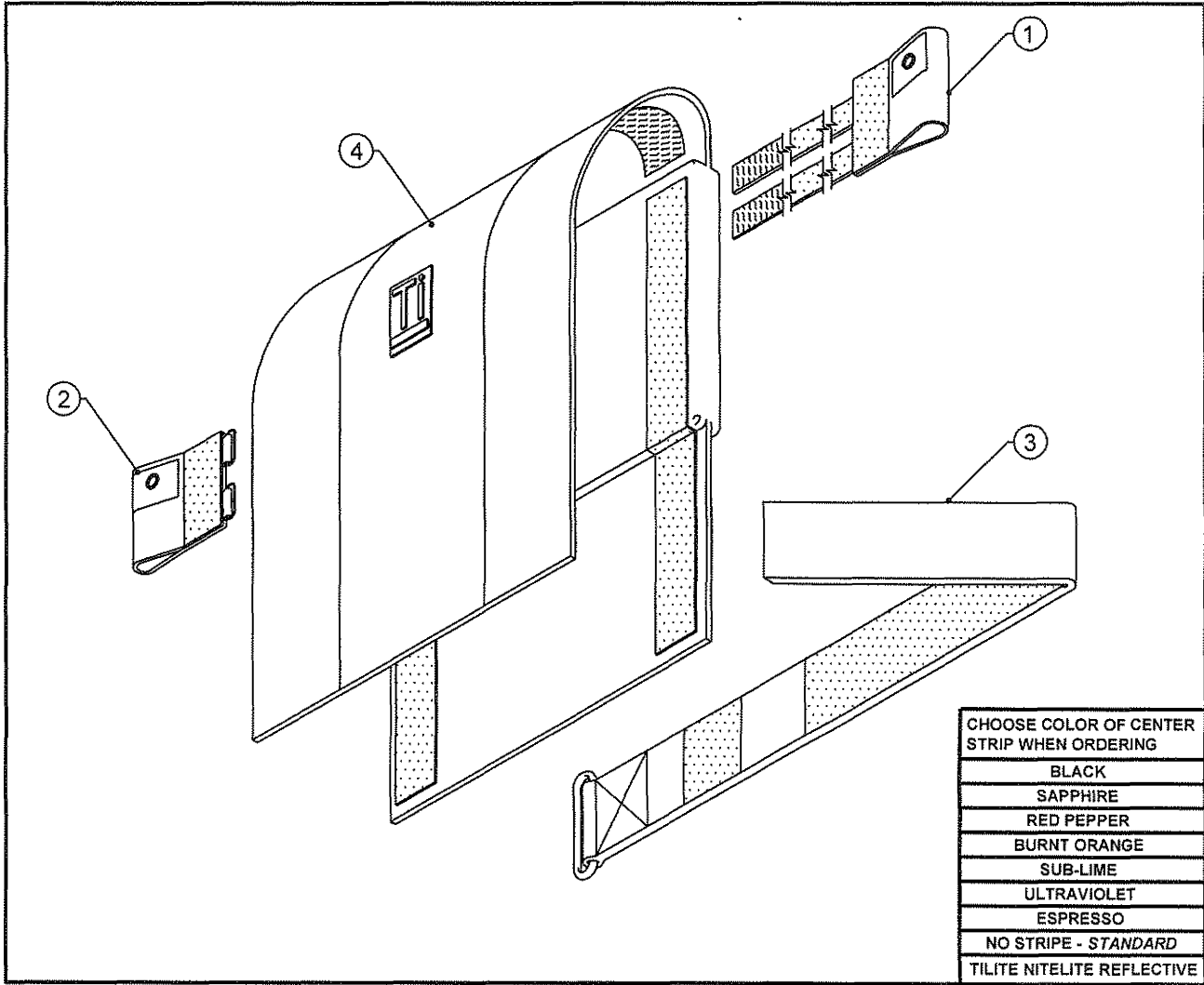

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP101503	BACKREST EAR, LEFT
3	1	TABLE TWO	UPHOLSTERY BACK STRAP (SEE NEXT PAGE)
4	1	TABLE THREE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-4		TABLE FOUR	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ XC, TX

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP101502
14"-15"	CP101504
16"-17"	CP101505
18"-19"	CP101506
20"	CP101507

TABLE TWO - UPHOLSTERY BACK STRAP

SEAT WIDTH	PART NUMBER
12"-13"	CP100492
14"-15"	CP100494
16"-17"	CP100496
18"-19"	CP100498
20"	CP101522

TABLE THREE - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	BACK HEIGHT 8"-12"	BACK HEIGHT 12"-16"	BACK HEIGHT 14"-18"	BACK HEIGHT 16"-20"
12"	CP101212	CP100833	CP100897	CP102893
13"	CP101213	CP100834	CP100868	CP102894
14"	CP101214	CP100835	CP100869	CP102895
15"	CP101215	CP100836	CP100870	CP102896
16"	CP101216	CP100837	CP100871	CP102897
17"	CP101217	CP100838	CP100872	CP102898
18"	CP101218	CP100839	CP100873	CP102899
19"	CP101570	CP101595	CP101567	CP102900
20"	CP101571	CP101596	CP101568	CP102901

TABLE FOUR - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	BACK HEIGHT 8"-12"	BACK HEIGHT 12"-16"	BACK HEIGHT 14"-18"	BACK HEIGHT 16"-20"
12"	AS103515	AS103524	AS103533	AS103542
13"	AS103516	AS103525	AS103534	AS103543
14"	AS103517	AS103526	AS103535	AS103544
15"	AS103518	AS103527	AS103536	AS103545
16"	AS103519	AS103528	AS103537	AS103546
17"	AS103520	AS103529	AS103538	AS103547
18"	AS103521	AS103530	AS103539	AS103548
19"	AS103522	AS103531	AS103540	AS103549
20"	AS103523	AS103532	AS103541	AS103550


NOTE:

8"-12" BACK HEIGHTS USE ONE UPHOLSTERY BACK STRAP

12"-16" BACK HEIGHTS USE TWO UPHOLSTERY BACK STRAPS


14"-18" BACK HEIGHTS USE THREE UPHOLSTERY BACK STRAPS

16"-20" BACK HEIGHTS USE FOUR UPHOLSTERY BACK STRAPS

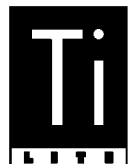

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ XC, TX

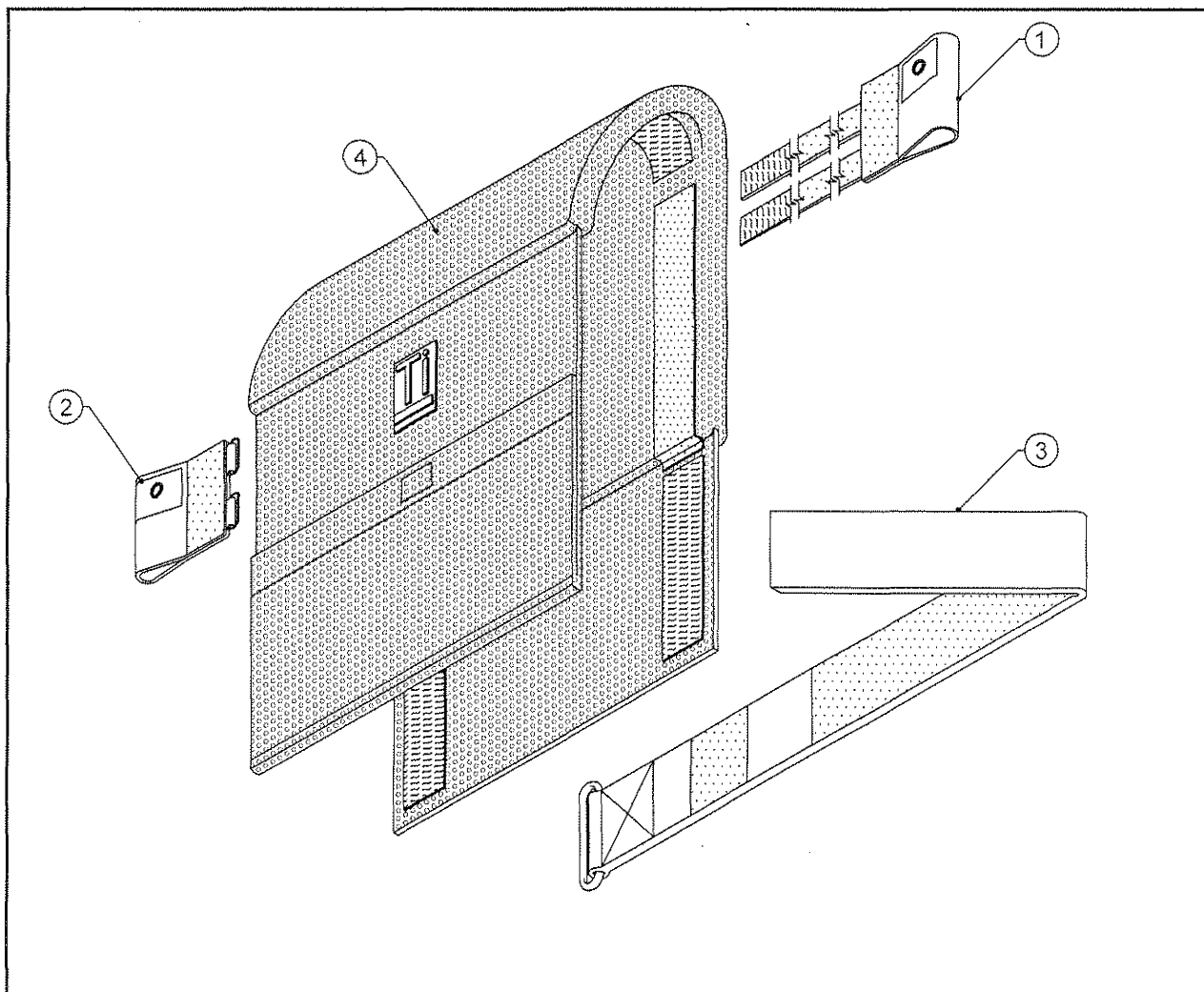

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP101503	BACKREST EAR, LEFT
3	1	TABLE TWO	UPHOLSTERY BACK STRAP (SEE NEXT PAGE)
4	1	TABLE THREE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-4		TABLE FOUR	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ADJUSTABLE HEIGHT PUSH HANDLES WITH FOLDING BACKREST BAR
TiLite™ XC, TX

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP101502
14"-15"	CP101504
16"-17"	CP101505
18"-19"	CP101506
20"	CP101507

TABLE TWO - UPHOLSTERY BACK STRAP

SEAT WIDTH	PART NUMBER
12"-13"	CP100492
14"-15"	CP100494
16"-17"	CP100496
18"-19"	CP100498
20"	CP101522

TABLE THREE - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	BACK HEIGHT 8"-12"	BACK HEIGHT 12"-16"	BACK HEIGHT 14"-18"	BACK HEIGHT 16"-20"
12"	CP107041	CP107050	CP107059	CP107068
13"	CP107042	CP107051	CP107060	CP107069
14"	CP107043	CP107052	CP107061	CP107070
15"	CP107044	CP107053	CP107062	CP107071
16"	CP107045	CP107054	CP107063	CP107072
17"	CP107046	CP107055	CP107064	CP107073
18"	CP107047	CP107056	CP107065	CP107074
19"	CP107048	CP107057	CP107066	CP107075
20"	CP107049	CP107058	CP107067	CP107076

TABLE FOUR - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	BACK HEIGHT 8"-12"	BACK HEIGHT 12"-16"	BACK HEIGHT 14"-18"	BACK HEIGHT 16"-20"
12"	AS115566	AS115575	AS115584	AS115593
13"	AS115567	AS115576	AS115585	AS115594
14"	AS115568	AS115577	AS115586	AS115595
15"	AS115569	AS115578	AS115587	AS115596
16"	AS115570	AS115579	AS115588	AS115597
17"	AS115571	AS115580	AS115589	AS115598
18"	AS115572	AS115581	AS115590	AS115599
19"	AS115573	AS115582	AS115591	AS115600
20"	AS115574	AS115583	AS115592	AS115601

NOTE:


8"-12" BACK HEIGHTS USE ONE UPHOLSTERY BACK STRAP
12"-16" BACK HEIGHTS USE TWO UPHOLSTERY BACK STRAPS
14"-18" BACK HEIGHTS USE THREE UPHOLSTERY BACK STRAPS
16"-20" BACK HEIGHTS USE FOUR UPHOLSTERY BACK STRAPS



TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR

TiLite™ XC, TX

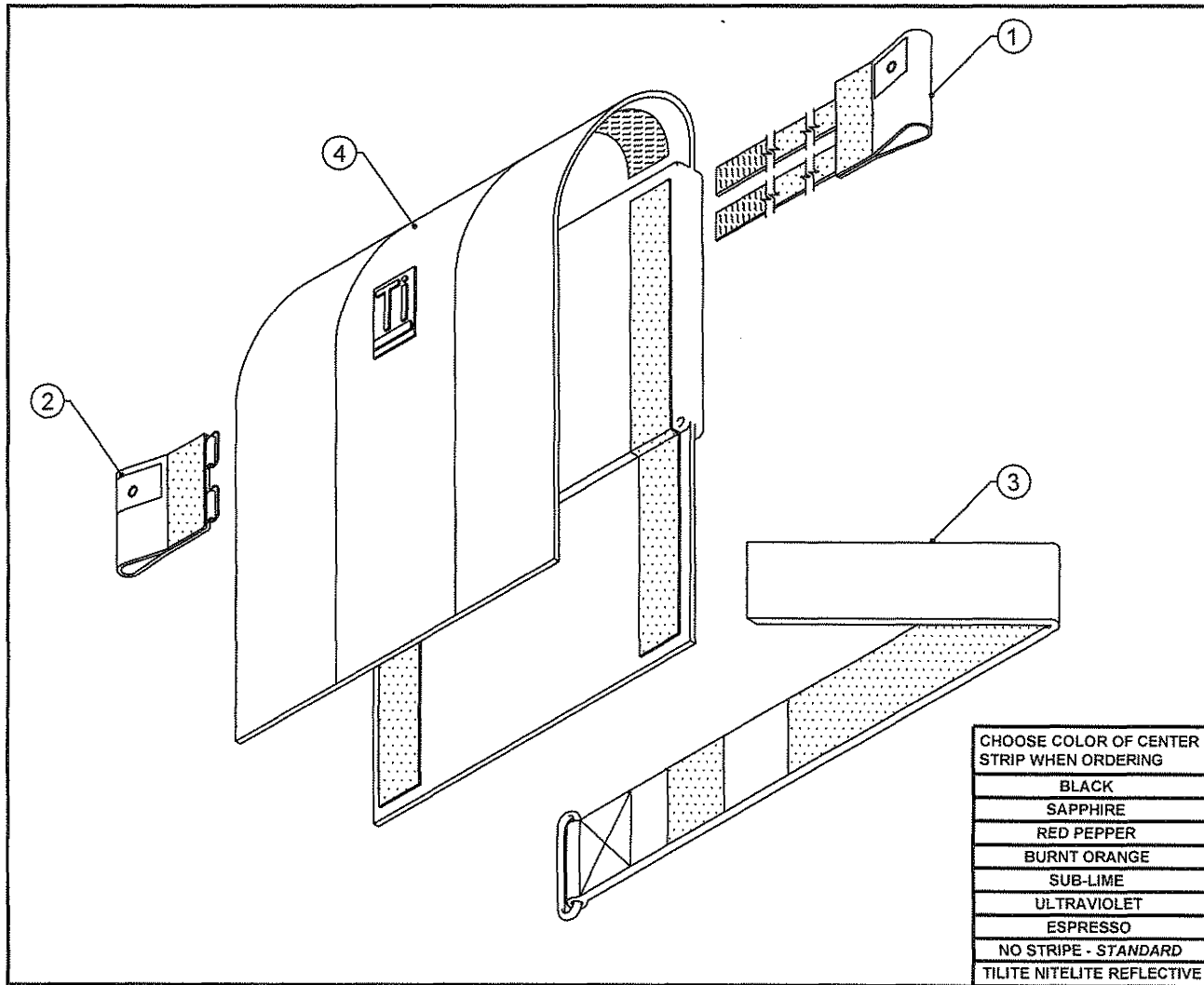

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP103705	BACKREST EAR, LEFT
3	1	TABLE TWO	UPHOLSTERY BACK STRAP (SEE NEXT PAGE)
4	1	TABLE THREE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-4		TABLE FOUR	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR
TiLite™ XC, TX

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP103707
14"-15"	CP103708
16"-17"	CP103709
18"-19"	CP103710
20"	CP103711

TABLE TWO - UPHOLSTERY BACK STRAP

SEAT WIDTH	PART NUMBER
12"-13"	CP100492
14"-15"	CP100494
16"-17"	CP100496
18"-19"	CP100498
20"	CP101522

TABLE THREE - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	14"-18.5" SEAT BACK HEIGHT	16"-20.5" SEAT BACK HEIGHT
12"	CP100897	CP102893
13"	CP100868	CP102894
14"	CP100869	CP102895
15"	CP100870	CP102896
16"	CP100871	CP102897
17"	CP100872	CP102898
18"	CP100873	CP102899
19"	CP101567	CP102900
20"	CP101568	CP102901

TABLE FOUR - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	14"-18.5" SEAT BACK HEIGHT	16"-20.5" SEAT BACK HEIGHT
12"	AS104121	AS104130
13"	AS104122	AS104131
14"	AS104123	AS104132
15"	AS104124	AS104133
16"	AS104125	AS104134
17"	AS104126	AS104135
18"	AS104127	AS104136
19"	AS104128	AS104137
20"	AS104129	AS104138


NOTE:

14"-18.5" BACK HEIGHTS USE THREE UPHOLSTERY BACK STRAPS

16"-20.5" BACK HEIGHTS USE FOUR UPHOLSTERY BACK STRAPS

BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR
TiLite™ XC, TX

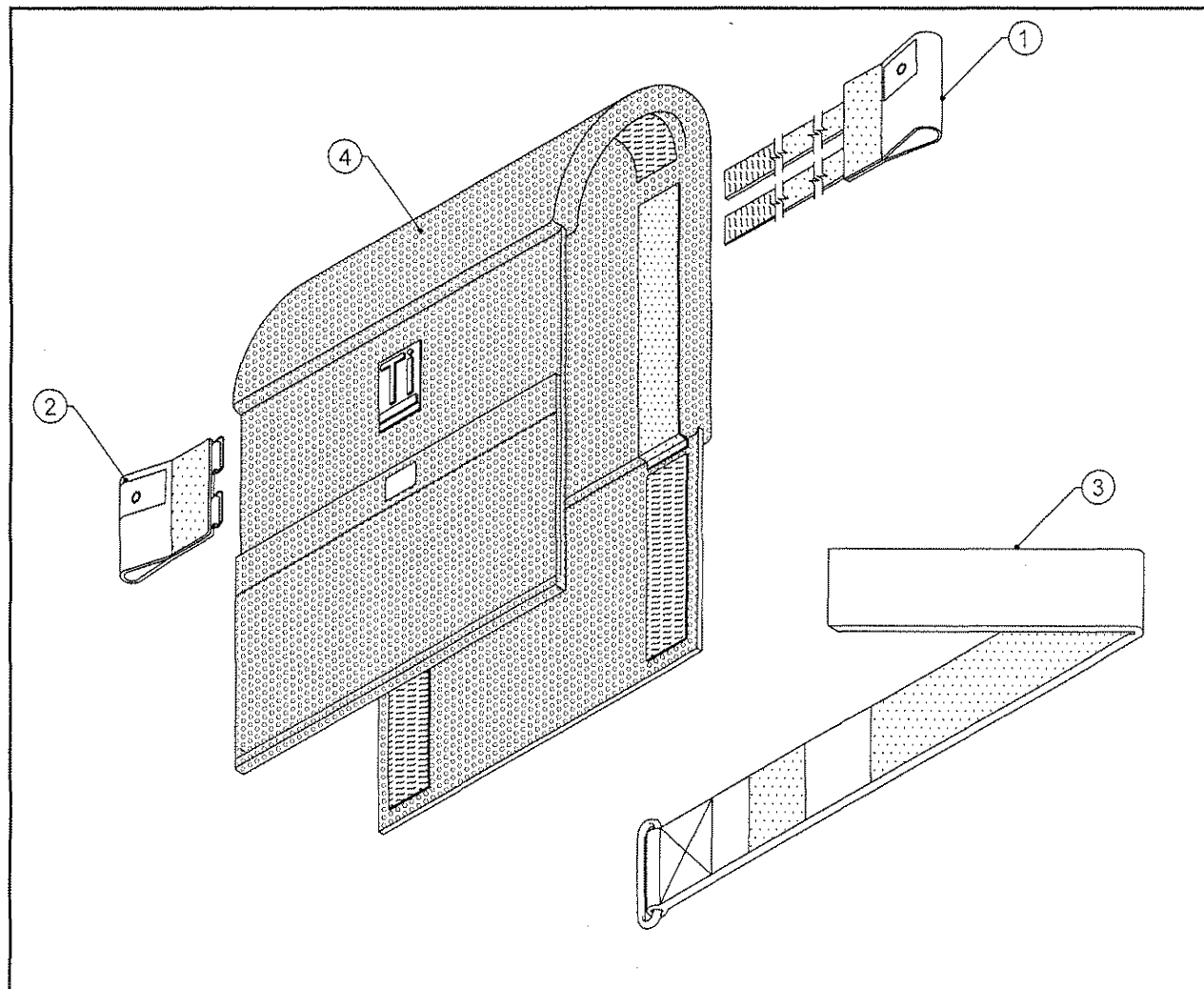

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	BACKREST EAR, RIGHT (SEE NEXT PAGE)
2	1	CP103705	BACKREST EAR, LEFT
3	1	TABLE TWO	UPHOLSTERY BACK STRAP (SEE NEXT PAGE)
4	1	TABLE THREE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
1-4		TABLE FOUR	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY (SEE NEXT PAGE)

TABLES AND ASSEMBLIES SHOWN ON NEXT PAGE

BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH FOLDING PUSH HANDLES AND FOLDING BACKREST BAR
TiLite™ XC, TX

TABLE ONE - BACKREST EAR, RIGHT

SEAT WIDTH	PART NUMBER
12"-13"	CP103707
14"-15"	CP103708
16"-17"	CP103709
18"-19"	CP103710
20"	CP103711

TABLE TWO - UPHOLSTERY BACK STRAP

SEAT WIDTH	PART NUMBER
12"-13"	CP100492
14"-15"	CP100494
16"-17"	CP100496
18"-19"	CP100498
20"	CP101522

TABLE THREE - BACK PAD TENSION ADJUSTABLE

SEAT WIDTH	14"-18.5" SEAT BACK HEIGHT	16"-20.5" SEAT BACK HEIGHT
12"	CP107059	CP107068
13"	CP107060	CP107069
14"	CP107061	CP107070
15"	CP107062	CP107071
16"	CP107063	CP107072
17"	CP107064	CP107073
18"	CP107065	CP107074
19"	CP107066	CP107075
20"	CP107067	CP107076

TABLE FOUR - TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLIES

SEAT WIDTH	14"-18.5" SEAT BACK HEIGHT	16"-20.5" SEAT BACK HEIGHT
12"	AS115791	AS115800
13"	AS115792	AS115801
14"	AS115793	AS115802
15"	AS115794	AS115803
16"	AS115795	AS115804
17"	AS115796	AS115805
18"	AS115797	AS115806
19"	AS115798	AS115807
20"	AS115799	AS115808


NOTE:

14"-18.5" BACK HEIGHTS USE THREE UPHOLSTERY BACK STRAPS

16"-20.5" BACK HEIGHTS USE FOUR UPHOLSTERY BACK STRAPS

VELCRO ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

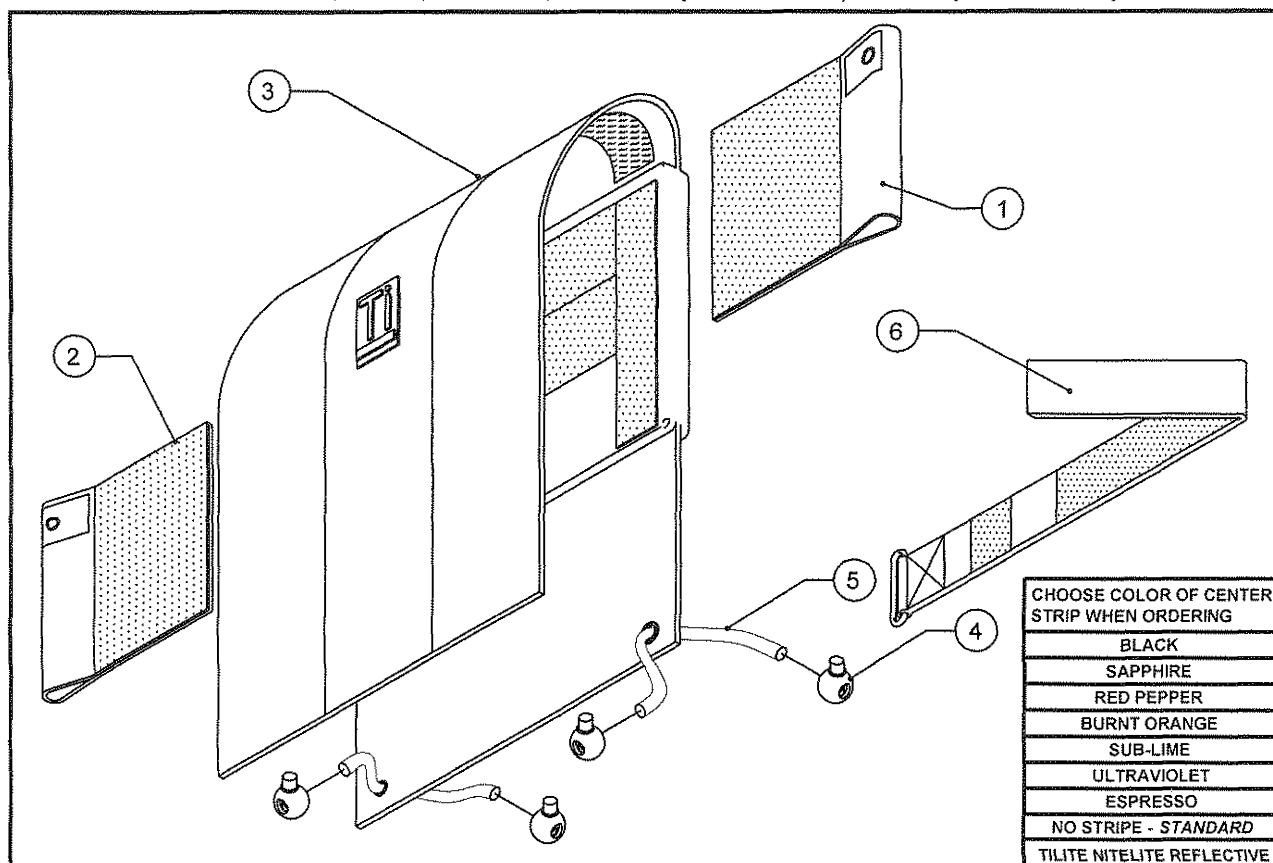

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE FOUR	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE FOUR	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FIVE	BACK PAD VELCRO ADJUSTABLE (SEE NEXT PAGE)
4	4	RW100026	BALL CORD LOCK, BLACK
5	2	RW100214	1/8" SHOCK CORD, BLACK (6")
6*	1-3	TABLE THREE	BACK STRAP (SEE NEXT PAGE)
1-6**		TABLE ONE	AERO X, 2GX VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY
1-3,6**		TABLE TWO	TX VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - VELCRO ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT Aero X, 2GX: 11"-13"	ADJ. HEIGHT Aero X, 2GX: 13.5"-16.5"	ADJ. HEIGHT Aero X, 2GX: 16.5"-19.5"	ADJ. HEIGHT W/PUSH HANDLES Aero X, 2GX: 13"-15"	ADJ. HEIGHT W/PUSH HANDLES Aero X, 2GX: 14"-17"	ADJ. HEIGHT W/PUSH HANDLES Aero X, 2GX: 17"-20"
12"	AS104634	AS104643	AS104652	AS104661	AS104670	AS104679
13"	AS104635	AS104644	AS104653	AS104662	AS104671	AS104680
14"	AS104636	AS104645	AS104654	AS104663	AS104672	AS104681
15"	AS104637	AS104646	AS104655	AS104664	AS104673	AS104682
16"	AS104638	AS104647	AS104656	AS104665	AS104674	AS104683
17"	AS104639	AS104648	AS104657	AS104666	AS104675	AS104684
18"	AS104640	AS104649	AS104658	AS104667	AS104676	AS104685
19"	AS104641	AS104650	AS104659	AS104668	AS104677	AS104686
20"	AS104642	AS104651	AS104660	AS104669	AS104678	AS104687



VELCRO ADJ. BACK UPHOLSTERY (CONT)

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO - TX VELCRO ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT TX: 13"-15"	ADJ. HEIGHT TX: 15.5"-18.5"	ADJ. HEIGHT TX: 18.5"-21.5"	ADJ. HEIGHT W/PUSH HANDLES TX: 15"-17"	ADJ. HEIGHT W/PUSH HANDLES TX: 16"-19"	ADJ. HEIGHT W/PUSH HANDLES TX: 19"-22"
12"	AS110513	AS110522	AS110531	AS110540	AS110550	AS110559
13"	AS110514	AS110523	AS110532	AS110541	AS110551	AS110560
14"	AS110515	AS110524	AS110533	AS110542	AS110552	AS110561
15"	AS110516	AS110525	AS110534	AS110543	AS110553	AS110562
16"	AS110517	AS110526	AS110535	AS110544	AS110554	AS110563
17"	AS110518	AS110527	AS110536	AS110545	AS110555	AS110564
18"	AS110519	AS110528	AS110537	AS110546	AS110556	AS110565
19"	AS110520	AS110529	AS110538	AS110547	AS110557	AS110566
20"	AS110521	AS110530	AS110539	AS110548	AS110558	AS110567

TABLE THREE - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 6

TX: 13"-15" / 15"-17" BACK HEIGHTS USE ONE 1" BACK STRAP
TX: 15.5"-18.5" / 16"-19" BACK HEIGHTS USE TWO 1" BACK STRAPS AND ONE 2" BACK STRAPS
TX: 18.5"-21.5" / 19"-22" BACK HEIGHTS USE ONE 1" BACK STRAP AND TWO 2" BACK STRAPS
2GX, Aero X: 11"-13" / 13"-15" BACK HEIGHTS USE NO STRAPS
2GX, Aero X: 13.5"-16.5" / 14"-17" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAPS
2GX, Aero X: 16.5"-19.5" / 17"-20" BACK HEIGHTS USE TWO 2" BACK STRAPS

TABLE FOUR - BACKREST EARS

CHAIR WIDTH	ADJ. HEIGHT TX: 13"-15" Aero X,2GX: 11"-13"	ADJ. HEIGHT TX: 15.5"-18.5" Aero X,2GX: 13.5"-16.5"	ADJ. HEIGHT TX: 18.5"-21.5" Aero X,2GX: 16.5"-19.5"	ADJ. HEIGHT W/PUSH HANDLES TX: 15"-17" Aero X,2GX: 13"-15"	ADJ. HEIGHT W/PUSH HANDLES TX: 16"-19" Aero X,2GX: 14"-17"	ADJ. HEIGHT W/PUSH HANDLES TX: 19"-22" Aero X,2GX: 17"-20"
12"-13" RIGHT	CP100450	CP100450	CP100450	CP100508	CP100508	CP100508
12"-13" LEFT	CP100514	CP100514	CP100514	CP100673	CP100673	CP100673
14"-18" RIGHT	CP100451	CP100451	CP100451	CP100509	CP100509	CP100509
14"-18" LEFT	CP100454	CP100454	CP100454	CP100507	CP100507	CP100507
19"-20" RIGHT	CP101526	CP101526	CP101526	CP101528	CP101528	CP101528
19"-20" LEFT	CP101525	CP101525	CP101525	CP101527	CP101527	CP101527


TABLE FIVE - BACK PAD VELCRO ADJUSTABLE

CHAIR WIDTH	ADJ. HEIGHT TX: 13"-15" Aero X,2GX: 11"-13"	ADJ. HEIGHT TX: 15.5"-18.5" Aero X,2GX: 13.5"-16.5"	ADJ. HEIGHT TX: 18.5"-21.5" Aero X,2GX: 16.5"-19.5"	ADJ. HEIGHT W/PUSH HANDLES TX: 15"-17" Aero X,2GX: 13"-15"	ADJ. HEIGHT W/PUSH HANDLES TX: 16"-19" Aero X,2GX: 14"-17"	ADJ. HEIGHT W/PUSH HANDLES TX: 19"-22" Aero X,2GX: 17"-20"
12"	CP103972	CP103981	CP103990	CP103972	CP103981	CP103990
13"	CP103973	CP103982	CP103991	CP103973	CP103982	CP103991
14"	CP103974	CP103983	CP103992	CP103974	CP103983	CP103992
15"	CP103975	CP103984	CP103993	CP103975	CP103984	CP103993
16"	CP103976	CP103985	CP103994	CP103976	CP103985	CP103994
17"	CP103977	CP103986	CP103995	CP103977	CP103986	CP103995
18"	CP103978	CP103987	CP103996	CP103978	CP103987	CP103996
19"	CP103979	CP103988	CP103997	CP103979	CP103988	CP103997
20"	CP103980	CP103989	CP103998	CP103980	CP103989	CP103998



VELCRO ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK WITH FOLDING PUSH HANDLES
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

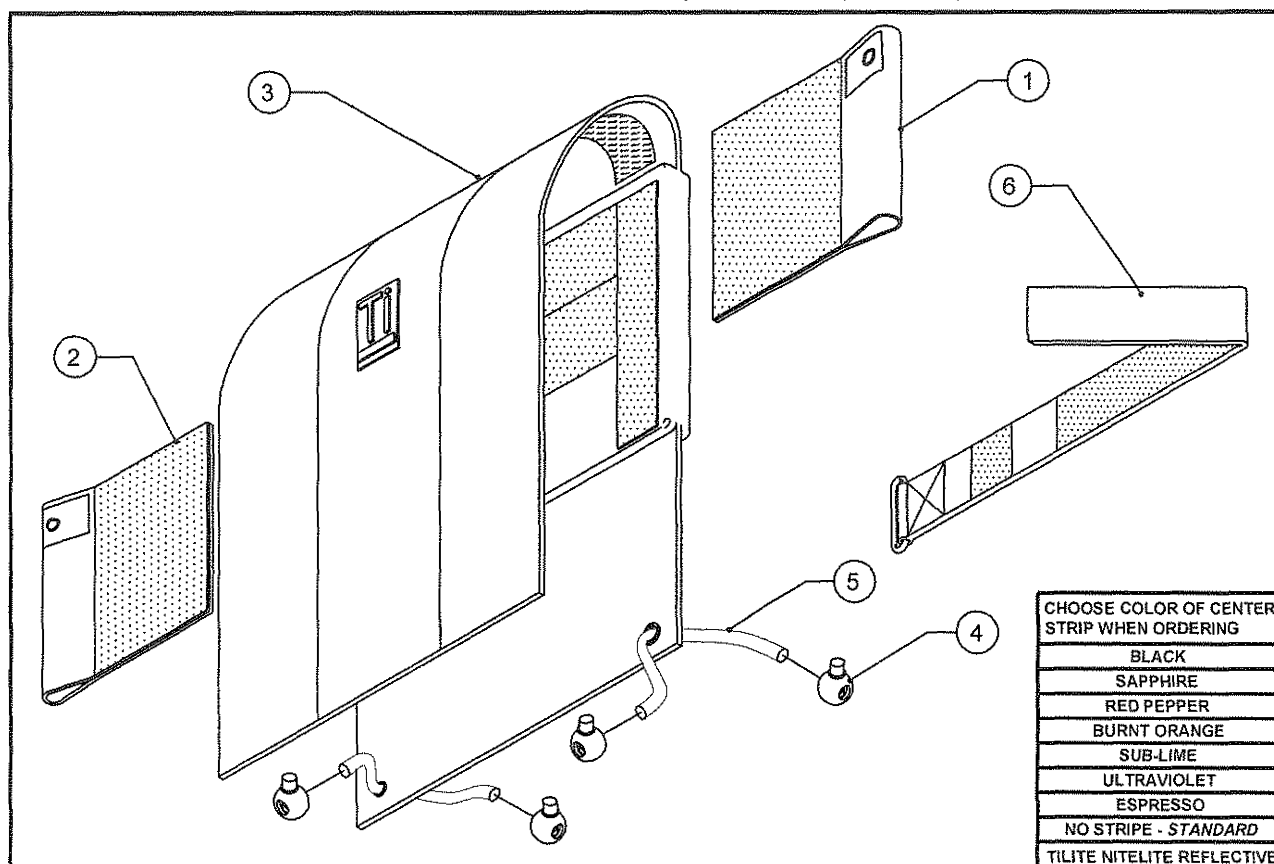

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD VELCRO ADJUSTABLE (SEE NEXT PAGE)
4	4	RW100026	BALL CORD LOCK, BLACK
5	2	RW100214	1/8" SHOCK CORD, BLACK (6")
6*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-6		TABLE ONE	AERO X, 2GX VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY
1-3,6		TABLE ONE	TX VELCRO ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - VELCRO ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	TX ADJ. HEIGHT 16.5"-18.5"	TX ADJ. HEIGHT 17.5"-20.5"	2GX, Aero X ADJ. HEIGHT 14.5"-16.5"	2GX, Aero X ADJ. HEIGHT 15.5"-18.5"	2GX, Aero X ADJ. HEIGHT 18.5"-21.5"
12"	AS104700	AS104709	AS104718	AS104700	AS104709
13"	AS104701	AS104710	AS104719	AS104701	AS104710
14"	AS104702	AS104711	AS104720	AS104702	AS104711
15"	AS104703	AS104712	AS104721	AS104703	AS104712
16"	AS104704	AS104713	AS104722	AS104704	AS104713
17"	AS104705	AS104714	AS104723	AS104705	AS104714
18"	AS104706	AS104715	AS104724	AS104706	AS104715
19"	AS104707	AS104716	AS104725	AS104707	AS104716
20"	AS104708	AS104717	AS104726	AS104708	AS104717

VELCRO ADJ. BACK UPHOLSTERY (CONT)

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK WITH FOLDING PUSH HANDLES
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 6

TX: 16.5"-18.5" BACK HEIGHTS USE ONE 1" BACK STRAP
TX: 17.5"-20.5" BACK HEIGHTS USE TWO 1" BACK STRAPS AND ONE 2" BACK STRAPS
2GX, Aero X: 14.5"-16.5" BACK HEIGHTS USE NO STRAPS
2GX, Aero X: 15.5"-18.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAPS
2GX, Aero X: 18.5"-21.5" BACK HEIGHTS USE TWO 2" BACK STRAPS

TABLE THREE - BACKREST EARS


CHAIR WIDTH	TX ADJ. HEIGHT 16.5"-18.5"	TX ADJ. HEIGHT 17.5"-20.5"	2GX, Aero X ADJ. HEIGHT 14.5"-16.5"	2GX, Aero X ADJ. HEIGHT 15.5"-18.5"	2GX, Aero X ADJ. HEIGHT 18.5"-21.5"
12"-13" RIGHT	CP103999	CP103999	CP103999	CP103999	CP103999
12"-13" LEFT	CP104000	CP104000	CP104000	CP104000	CP104000
14"-18" RIGHT	CP104001	CP104001	CP104001	CP104001	CP104001
14"-18" LEFT	CP104002	CP104002	CP104002	CP104002	CP104002
19"-20" RIGHT	CP104003	CP104003	CP104003	CP104003	CP104003
19"-20" LEFT	CP104004	CP104004	CP104004	CP104004	CP104004

TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	TX 16.5"-18.5" SEAT BACK HEIGHT	TX 17.5"-20.5" SEAT BACK HEIGHT	2GX, Aero X 14.5"-16.5" SEAT BACK HEIGHT	2GX, Aero X 15.5"-18.5" SEAT BACK HEIGHT	2GX, Aero X 18.5"-21.5" SEAT BACK HEIGHT
12"	CP103981	CP103990	CP103972	CP103981	CP103990
13"	CP103982	CP103991	CP103973	CP103982	CP103991
14"	CP103983	CP103992	CP103974	CP103983	CP103992
15"	CP103984	CP103993	CP103975	CP103984	CP103993
16"	CP103985	CP103994	CP103976	CP103985	CP103994
17"	CP103986	CP103995	CP103977	CP103986	CP103995
18"	CP103987	CP103996	CP103978	CP103987	CP103996
19"	CP103988	CP103997	CP103979	CP103988	CP103997
20"	CP103989	CP103998	CP103980	CP103989	CP103998

TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

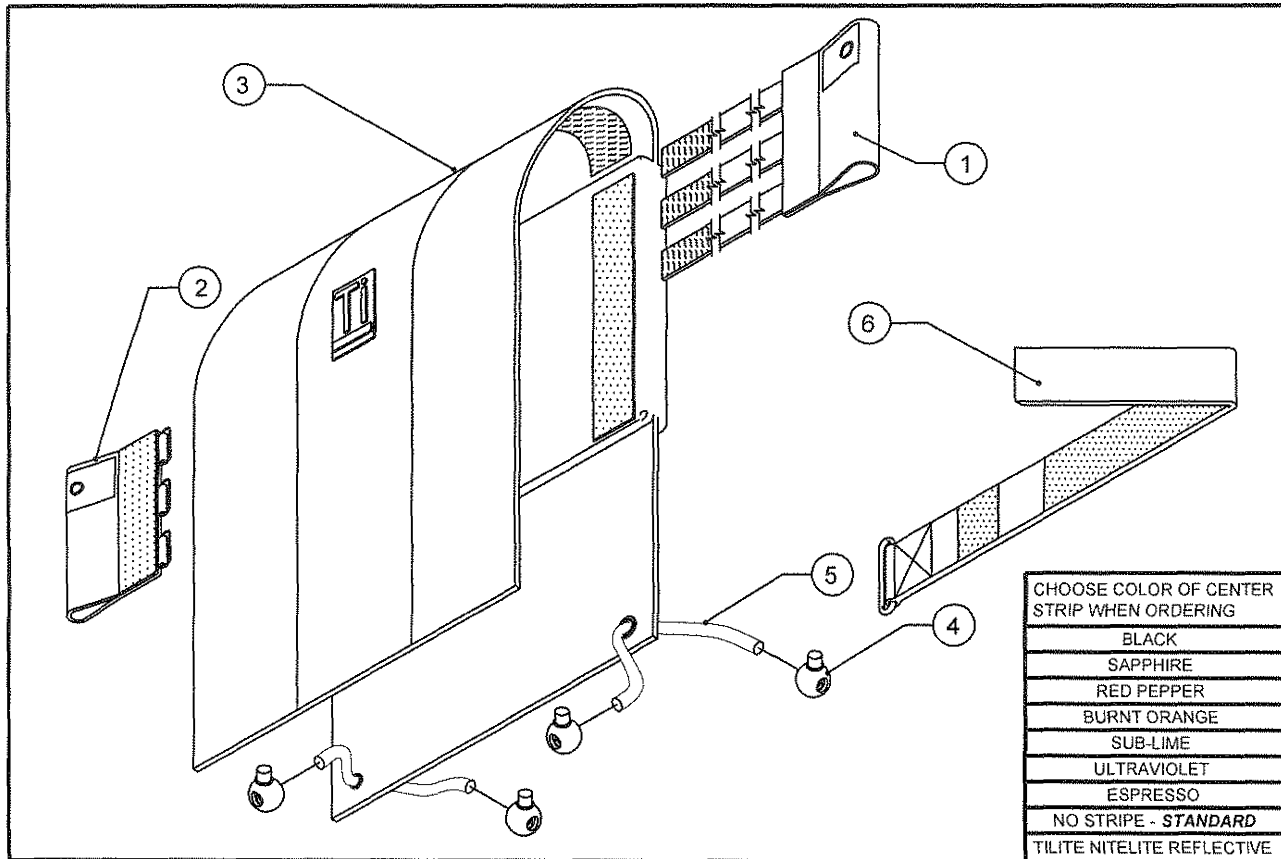

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE FOUR	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE FOUR	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FIVE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4	4	RW100026	BALL CORD LOCK, BLACK
5	2	RW100214	1/8" SHOCK CORD, BLACK
6*	1-3	TABLE THREE	BACK STRAP (SEE NEXT PAGE)
1-6**		TABLE ONE	AERO X, 2GX TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY
1-3,6**		TABLE TWO	TX TENSION ADJUSTABLE BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HEIGHT 11.5"-13.5"	STANDARD ADJ. HEIGHT 14"-17"	STANDARD ADJ. HEIGHT 17"-20"	ADJ. HT 11.5"-13.5" W/PUSH HANDLES	ADJ. HT 14"-17" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"	AS103724	AS103733	AS103742	AS103751	AS103760	AS103769
13"	AS103725	AS103734	AS103743	AS103752	AS103761	AS103770
14"	AS103726	AS103735	AS103744	AS103753	AS103762	AS103771
15"	AS103727	AS103736	AS103745	AS103754	AS103763	AS103772
16"	AS103728	AS103737	AS103746	AS103755	AS103764	AS103773
17"	AS103729	AS103738	AS103747	AS103756	AS103765	AS103774
18"	AS103730	AS103739	AS103748	AS103757	AS103766	AS103775
19"	AS103731	AS103740	AS103749	AS103758	AS103767	AS103776
20"	AS103732	AS103741	AS103750	AS103759	AS103768	AS103777



TENSION ADJ. BACK UPHOLSTERY (CONT)

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2)

TABLE TWO - TX TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT TX: 11.5"-13.5"	ADJ. HEIGHT TX: 14"-17"	ADJ. HEIGHT TX: 17"-20"	ADJ. HEIGHT W/PUSH HANDLES TX: 11.5"-13.5"	ADJ. HEIGHT W/PUSH HANDLES TX: 14"-17"	ADJ. HEIGHT W/PUSH HANDLES TX: 17"-20"
12"	AS110588	AS110595	AS110604	AS110613	AS110622	AS110631
13"	AS110587	AS110596	AS110605	AS110614	AS110623	AS110632
14"	AS110588	AS110597	AS110606	AS110615	AS110624	AS110633
15"	AS110589	AS110598	AS110607	AS110616	AS110625	AS110634
16"	AS110590	AS110599	AS110608	AS110617	AS110626	AS110635
17"	AS110591	AS110600	AS110609	AS110618	AS110627	AS110636
18"	AS110592	AS110601	AS110610	AS110619	AS110628	AS110637
19"	AS110593	AS110602	AS110611	AS110620	AS110629	AS110638
20"	AS110594	AS110603	AS110612	AS110621	AS110630	AS110639

TABLE THREE - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 6

TX: 11.5"-13.5" BACK HEIGHTS USE ONE 1" BACK STRAP
TX: 14"-17" BACK HEIGHTS USE TWO 1" BACK STRAPS AND ONE 2" BACK STRAPS
TX: 17"-20" BACK HEIGHTS USE ONE 1" BACK STRAP AND TWO 2" BACK STRAPS
2GX, Aero X: 11.5"-13.5" BACK HEIGHTS USE NO STRAPS
2GX, Aero X: 14"-17" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAPS
2GX, Aero X: 17"-20" BACK HEIGHTS USE TWO 2" BACK STRAPS

TABLE FOUR - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HEIGHT 11.5"-13.5"	STANDARD ADJ. HEIGHT 14"-17"	STANDARD ADJ. HEIGHT 17"-20"	ADJ. HT 11.5"-13.5" W/PUSH HANDLES	ADJ. HT 14"-17" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"-13" RIGHT	CP100049	CP100049	CP100049	CP100323	CP100323	CP100323
12"-13" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
14"-15" RIGHT	CP100264	CP100264	CP100264	CP100324	CP100324	CP100324
14"-15" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
16"-17" RIGHT	CP100265	CP100265	CP100265	CP100325	CP100325	CP100325
16"-17" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
18"-19" RIGHT	CP100266	CP100266	CP100266	CP100326	CP100326	CP100326
18"-19" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
20" RIGHT	CP101508	CP101508	CP101508	CP101494	CP101494	CP101494
20" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322


TABLE FIVE - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	14"-16" SEAT BACK HEIGHT	15.5"-18.5" SEAT BACK HEIGHT	18.5"-21.5" SEAT BACK HEIGHT
12"	CP100799	CP100584	CP103466
13"	CP100800	CP100585	CP103467
14"	CP100801	CP100586	CP103468
15"	CP100802	CP100587	CP103469
16"	CP100803	CP100588	CP103470
17"	CP100804	CP100589	CP103471
18"	CP100805	CP100590	CP103472
19"	CP101553	CP101555	CP103473
20"	CP101554	CP101556	CP103474



BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

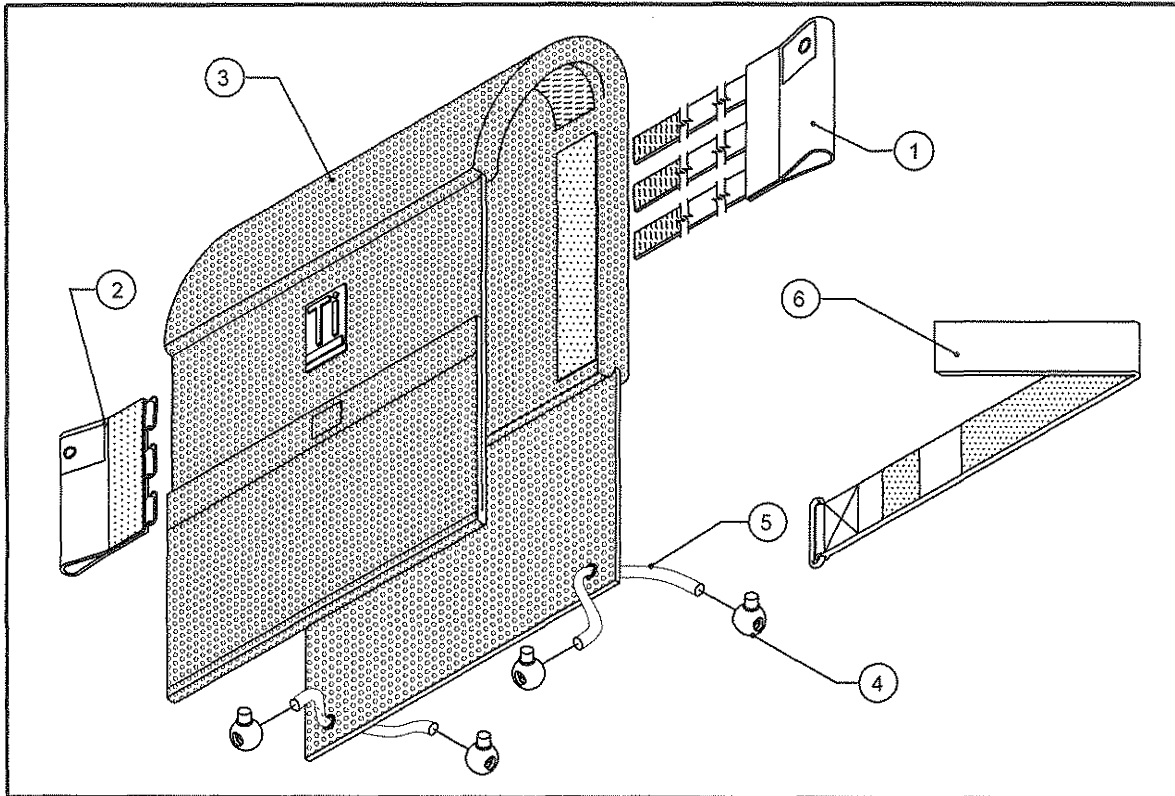

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE FOUR	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE FOUR	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FIVE	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4	4	RW100026	BALL CORD LOCK, BLACK
5	2	RW100214	1/8" SHOCK CORD, BLACK
6*	1-3	TABLE THREE	BACK STRAP (SEE NEXT PAGE)
1-6**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY W/ PUSH ASSEMBLY
1-3,6**		TABLE TWO	TX TENSION ADJUSTABLE, BACK UPHOLSTERY W/ PUSH ASSEMBLY
1-6**		TABLE ONE	TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY
1-3,6**		TABLE TWO	TX TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	STANDARD ADJ. HEIGHT 11.5"-13.5"	STANDARD ADJ. HEIGHT 14"-17"	STANDARD ADJ. HEIGHT 17"-20"	ADJ. HT 11.5"-13.5" W/PUSH HANDLES	ADJ. HT 14"-17" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"	AS115602	AS115611	AS115620	AS115629	AS115638	AS115647
13"	AS115603	AS115612	AS115621	AS115630	AS115639	AS115648
14"	AS115604	AS115613	AS115622	AS115631	AS115640	AS115649
15"	AS115605	AS115614	AS115623	AS115632	AS115641	AS115650
16"	AS115606	AS115615	AS115624	AS115633	AS115642	AS115651
17"	AS115607	AS115616	AS115625	AS115634	AS115643	AS115652
18"	AS115608	AS115617	AS115626	AS115635	AS115644	AS115653
19"	AS115609	AS115618	AS115627	AS115636	AS115645	AS115654
20"	AS115610	AS115619	AS115628	AS115637	AS115646	AS115655



BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT)

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2)

TABLE TWO - TX TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	ADJ. HEIGHT TX: 11.5"-13.5"	ADJ. HEIGHT TX: 14"-17"	ADJ. HEIGHT TX: 17"-20"	ADJ. HEIGHT W/PUSH HANDLES TX: 11.5"-13.5"	ADJ. HEIGHT W/PUSH HANDLES TX: 14"-17"	ADJ. HEIGHT W/PUSH HANDLES TX: 17"-20"
12"	AS115656	AS115665	AS115674	AS115683	AS115692	AS115701
13"	AS115657	AS115666	AS115675	AS115684	AS115693	AS115702
14"	AS115658	AS115667	AS115676	AS115685	AS115694	AS115703
15"	AS115659	AS115668	AS115677	AS115686	AS115695	AS115704
16"	AS115660	AS115669	AS115678	AS115687	AS115696	AS115705
17"	AS115661	AS115670	AS115679	AS115688	AS115697	AS115706
18"	AS115662	AS115671	AS115680	AS115689	AS115698	AS115707
19"	AS115663	AS115672	AS115681	AS115690	AS115699	AS115708
20"	AS115664	AS115673	AS115682	AS115691	AS115700	AS115709

TABLE THREE - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 6

TX: 11.5"-13.5" BACK HEIGHTS USE ONE 1" BACK STRAP
TX: 14"-17" BACK HEIGHTS USE TWO 1" BACK STRAPS AND ONE 2" BACK STRAPS
TX: 17"-20" BACK HEIGHTS USE ONE 1" BACK STRAP AND TWO 2" BACK STRAPS
2GX, Aero X: 11.5"-13.5" BACK HEIGHTS USE NO STRAPS
2GX, Aero X: 14"-17" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAPS
2GX, Aero X: 17"-20" BACK HEIGHTS USE TWO 2" BACK STRAPS

TABLE FOUR - BACKREST EARS

CHAIR WIDTH	STANDARD ADJ. HEIGHT 11.5"-13.5"	STANDARD ADJ. HEIGHT 14"-17"	STANDARD ADJ. HEIGHT 17"-20"	ADJ. HT 11.5"-13.5" W/PUSH HANDLES	ADJ. HT 14"-17" W/PUSH HANDLES	ADJ. HT 17"-20" W/PUSH HANDLES
12"-13" RIGHT	CP100049	CP100049	CP100049	CP100323	CP100323	CP100323
12"-13" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
14"-15" RIGHT	CP100264	CP100264	CP100264	CP100324	CP100324	CP100324
14"-15" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
16"-17" RIGHT	CP100265	CP100265	CP100265	CP100325	CP100325	CP100325
16"-17" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
18"-19" RIGHT	CP100266	CP100266	CP100266	CP100326	CP100326	CP100326
18"-19" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322
20" RIGHT	CP101508	CP101508	CP101508	CP101494	CP101494	CP101494
20" LEFT	CP100050	CP100050	CP100050	CP100322	CP100322	CP100322

TABLE FIVE - BACK PAD TENSION ADJUSTABLE


CHAIR WIDTH	14"-16" SEAT BACK HEIGHT	16.5"-18.5" SEAT BACK HEIGHT	18.5"-21.5" SEAT BACK HEIGHT
12"	CP107023	CP107032	CP107077
13"	CP107024	CP107033	CP107078
14"	CP107025	CP107034	CP107079
15"	CP107026	CP107035	CP107080
16"	CP107027	CP107036	CP107081
17"	CP107028	CP107037	CP107082
18"	CP107029	CP107038	CP107083
19"	CP107030	CP107039	CP107084
20"	CP107031	CP107040	CP107085



TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK WITH FOLDING PUSH HANDLES

TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

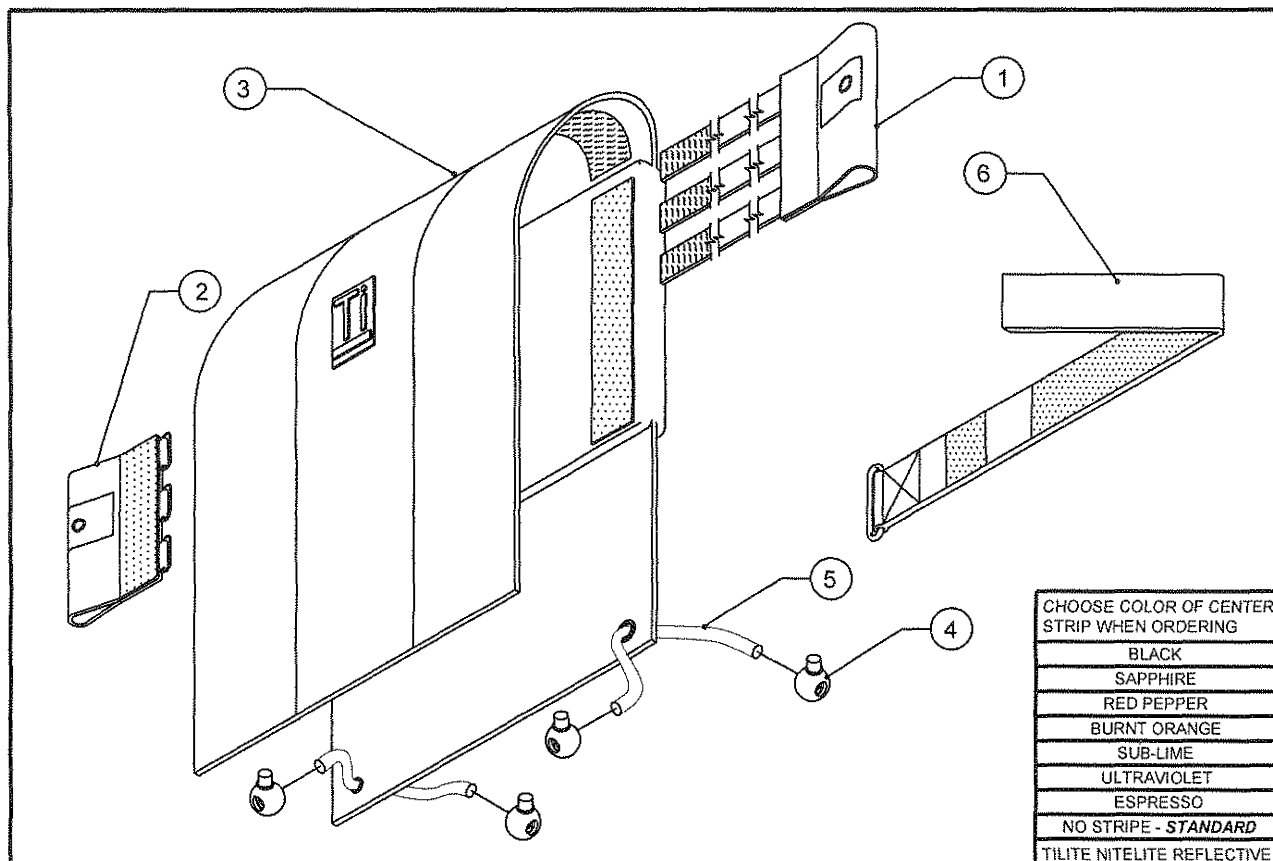

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4	4	RW100026	BALL CORD LOCK, BLACK
5	2	RW100214	1/8" SHOCK CORD, BLACK
6*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-6		TABLE ONE	AERO X, 2GX TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY
1-3,6		TABLE ONE	TX TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	TX ADJ. HEIGHT 16.5"-18.5"	TX ADJ. HEIGHT 17.5"-20.5"	2GX, Aero X ADJ. HEIGHT 14.5"-16.5"	2GX, Aero X ADJ. HEIGHT 15.5"-18.5"	2GX, Aero X ADJ. HEIGHT 18.5"-21.5"
12"	AS103856	AS103865	AS103847	AS103856	AS103865
13"	AS103857	AS103866	AS103848	AS103857	AS103866
14"	AS103858	AS103867	AS103849	AS103858	AS103867
15"	AS103859	AS103868	AS103850	AS103859	AS103868
16"	AS103860	AS103869	AS103851	AS103860	AS103869
17"	AS103861	AS103870	AS103852	AS103861	AS103870
18"	AS103862	AS103871	AS103853	AS103862	AS103871
19"	AS103863	AS103872	AS103854	AS103863	AS103872
20"	AS103864	AS103873	AS103855	AS103864	AS103873

TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK WITH FOLDING PUSH HANDLES
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 6

TX: 16.5"-18.5" BACK HEIGHTS USE ONE 1" BACK STRAP
TX: 17.5"-20.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP
2GX, Aero X: 14.5"-16.5" BACK HEIGHTS USE NO STRAPS
2GX, Aero X: 15.5"-18.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAPS
2GX, Aero X: 18.5"-21.5" BACK HEIGHTS USE TWO 2" BACK STRAPS

TABLE THREE - BACKREST EARS

CHAIR WIDTH	TX ADJ. HEIGHT 16.5"-18.5"	TX ADJ. HEIGHT 17.5"-20.5"	2GX, Aero X ADJ. HEIGHT 14.5"-16.5"	2GX, Aero X ADJ. HEIGHT 15.5"-18.5"	2GX, Aero X ADJ. HEIGHT 18.5"-21.5"
12"-13" RIGHT	CP103586	CP103586	CP103586	CP103586	CP103586
12"-13" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
14"-15" RIGHT	CP103587	CP103587	CP103587	CP103587	CP103587
14"-15" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
16"-17" RIGHT	CP103588	CP103588	CP103588	CP103588	CP103588
16"-17" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
18"-19" RIGHT	CP103589	CP103589	CP103589	CP103589	CP103589
18"-19" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
20" RIGHT	CP103590	CP103590	CP103590	CP103590	CP103590
20" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	TX 16.5"-18.5" SEAT BACK HEIGHT	TX 17.5"-20.5" SEAT BACK HEIGHT	2GX, Aero X 14.5"-16.5" SEAT BACK HEIGHT	2GX, Aero X 15.5"-18.5" SEAT BACK HEIGHT	2GX, Aero X 18.5"-21.5" SEAT BACK HEIGHT
12"	CP100584	CP103466	CP100799	CP100584	CP103466
13"	CP100585	CP103467	CP100800	CP100585	CP103467
14"	CP100586	CP103468	CP100801	CP100586	CP103468
15"	CP100587	CP103469	CP100802	CP100587	CP103469
16"	CP100588	CP103470	CP100803	CP100588	CP103470
17"	CP100589	CP103471	CP100804	CP100589	CP103471
18"	CP100590	CP103472	CP100805	CP100590	CP103472
19"	CP101555	CP103473	CP101553	CP101555	CP103473
20"	CP101556	CP103474	CP101554	CP101556	CP103474



BREATHABLE TENSION ADJUSTABLE BACK UPHOLSTERY

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK WITH FOLDING PUSH HANDLES
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

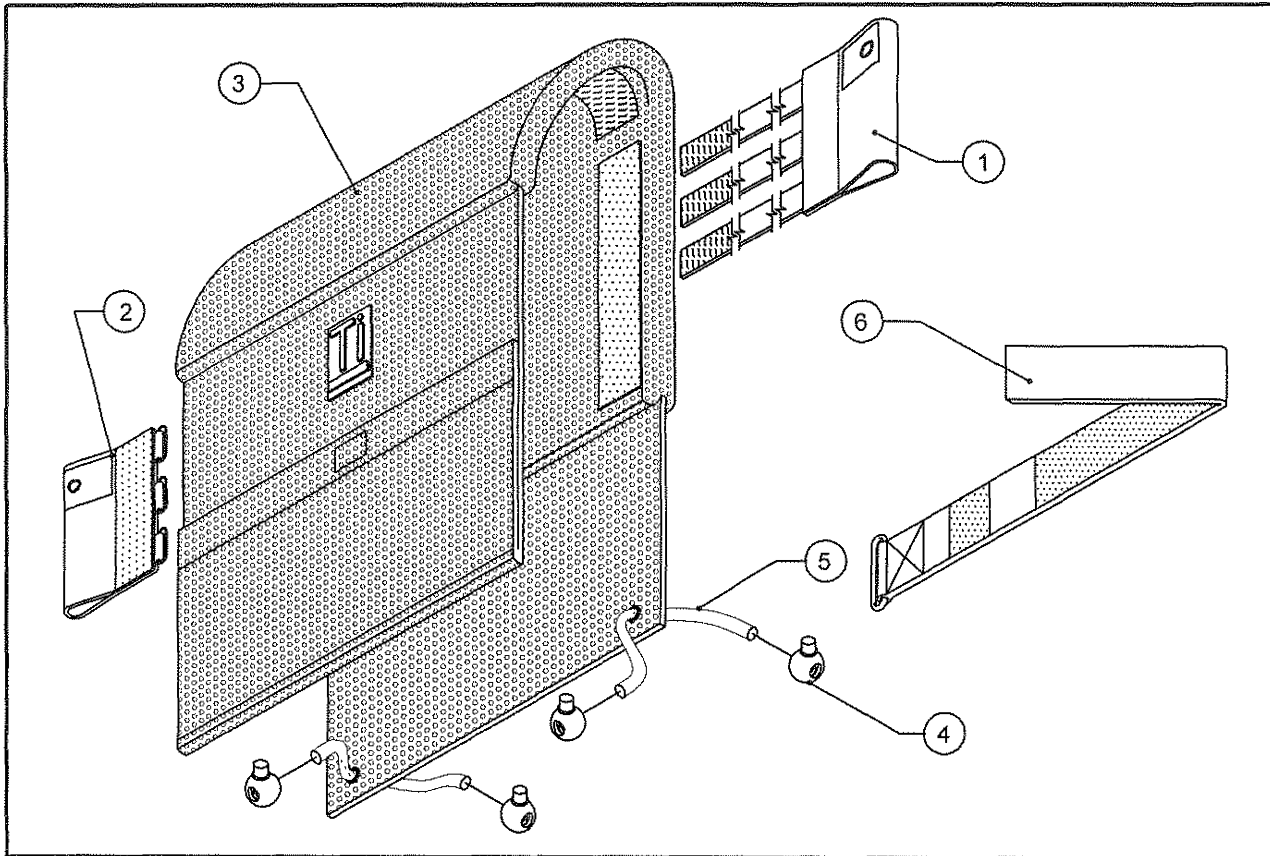

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE THREE	BACKREST EAR RIGHT (SEE NEXT PAGE)
2	1	TABLE THREE	BACKREST EAR LEFT (SEE NEXT PAGE)
3	1	TABLE FOUR	BACK PAD TENSION ADJUSTABLE (SEE NEXT PAGE)
4	4	RW100026	BALL CORD LOCK, BLACK
5	2	RW100214	1/8" SHOCK CORD, BLACK
6*	1-3	TABLE TWO	BACK STRAP (SEE NEXT PAGE)
1-6		TABLE ONE	AERO X, 2GX TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY
1-3,6		TABLE ONE	TX TENSION ADJUSTABLE, BACK UPHOLSTERY ASSEMBLY

TABLE ONE - TENSION ADJUSTABLE BACKREST ASSEMBLIES

CHAIR WIDTH	TX ADJ. HEIGHT 16.5"-18.5"	TX ADJ. HEIGHT 17.5"-20.5"	2GX, Aero X ADJ. HEIGHT 14.5"-16.5"	2GX, Aero X ADJ. HEIGHT 15.5"-18.5"	2GX, Aero X ADJ. HEIGHT 18.5"-21.5"
12"	AS115737	AS115746	AS115728	AS115737	AS115746
13"	AS115738	AS115747	AS115729	AS115738	AS115747
14"	AS115739	AS115748	AS115730	AS115739	AS115748
15"	AS115740	AS115749	AS115731	AS115740	AS115749
16"	AS115741	AS115750	AS115732	AS115741	AS115750
17"	AS115742	AS115751	AS115733	AS115742	AS115751
18"	AS115743	AS115752	AS115734	AS115743	AS115752
19"	AS115744	AS115753	AS115735	AS115744	AS115753
20"	AS115745	AS115754	AS115736	AS115745	AS115754

BREATHABLE TENSION ADJ. BACK UPHOLSTERY (CONT.)

FOR USE WITH ANGLE ADJUSTABLE FOLD DOWN BACK WITH FOLDING PUSH HANDLES
TiLite™ TX, 2GX, Aero X, Aero X (SERIES 2), 2GX (SERIES 2)

TABLE TWO - BACK STRAPS

BACKSTRAP WIDTH	12"-13" CHAIR WIDTH	14"-15" CHAIR WIDTH	16"-17" CHAIR WIDTH	18"-19" CHAIR WIDTH	20" CHAIR WIDTH
1" WIDE	CP101517	CP101518	CP101519	CP101520	CP101521
2" WIDE	CP100492	CP100494	CP100496	CP100498	CP101522

*NOTE: FOR ITEM 6

TX: 16.5"-18.5" BACK HEIGHTS USE ONE 1" BACK STRAP
TX: 17.5"-20.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAP
2GX, Aero X: 14.5"-16.5" BACK HEIGHTS USE NO STRAPS
2GX, Aero X: 15.5"-18.5" BACK HEIGHTS USE ONE 1" BACK STRAP AND ONE 2" BACK STRAPS
2GX, Aero X: 18.5"-21.5" BACK HEIGHTS USE TWO 2" BACK STRAPS

TABLE THREE - BACKREST EARS

CHAIR WIDTH	TX ADJ. HEIGHT 16.5"-18.5"	TX ADJ. HEIGHT 17.5"-20.5"	2GX, Aero X ADJ. HEIGHT 14.5"-16.5"	2GX, Aero X ADJ. HEIGHT 15.5"-18.5"	2GX, Aero X ADJ. HEIGHT 18.5"-21.5"
12"-13" RIGHT	CP103586	CP103586	CP103586	CP103586	CP103586
12"-13" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
14"-15" RIGHT	CP103587	CP103587	CP103587	CP103587	CP103587
14"-15" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
16"-17" RIGHT	CP103588	CP103588	CP103588	CP103588	CP103588
16"-17" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
18"-19" RIGHT	CP103589	CP103589	CP103589	CP103589	CP103589
18"-19" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585
20" RIGHT	CP103590	CP103590	CP103590	CP103590	CP103590
20" LEFT	CP103585	CP103585	CP103585	CP103585	CP103585


TABLE FOUR - BACK PAD TENSION ADJUSTABLE

CHAIR WIDTH	TX 16.5"-18.5" SEAT BACK HEIGHT	TX 17.5"-20.5" SEAT BACK HEIGHT	2GX, Aero X 14.5"-16.5" SEAT BACK HEIGHT	2GX, Aero X 15.5"-18.5" SEAT BACK HEIGHT	2GX, Aero X 18.5"-21.5" SEAT BACK HEIGHT
12"	CP107032	CP107077	CP107023	CP107032	CP107077
13"	CP107033	CP107078	CP107024	CP107033	CP107078
14"	CP107034	CP107079	CP107025	CP107034	CP107079
15"	CP107035	CP107080	CP107026	CP107035	CP107080
16"	CP107036	CP107081	CP107027	CP107036	CP107081
17"	CP107037	CP107082	CP107028	CP107037	CP107082
18"	CP107038	CP107083	CP107029	CP107038	CP107083
19"	CP107039	CP107084	CP107030	CP107039	CP107084
20"	CP107040	CP107085	CP107031	CP107040	CP107085



CORBAC BACKREST UPHOLSTERY

TiLite™ ZR, ZRA, TR (SERIES 2), YR (SERIES 2), Aero Z,
Aero R, ZR(SERIES 2), ZRA(SERIES 2), AERO Z (SERIES 2)

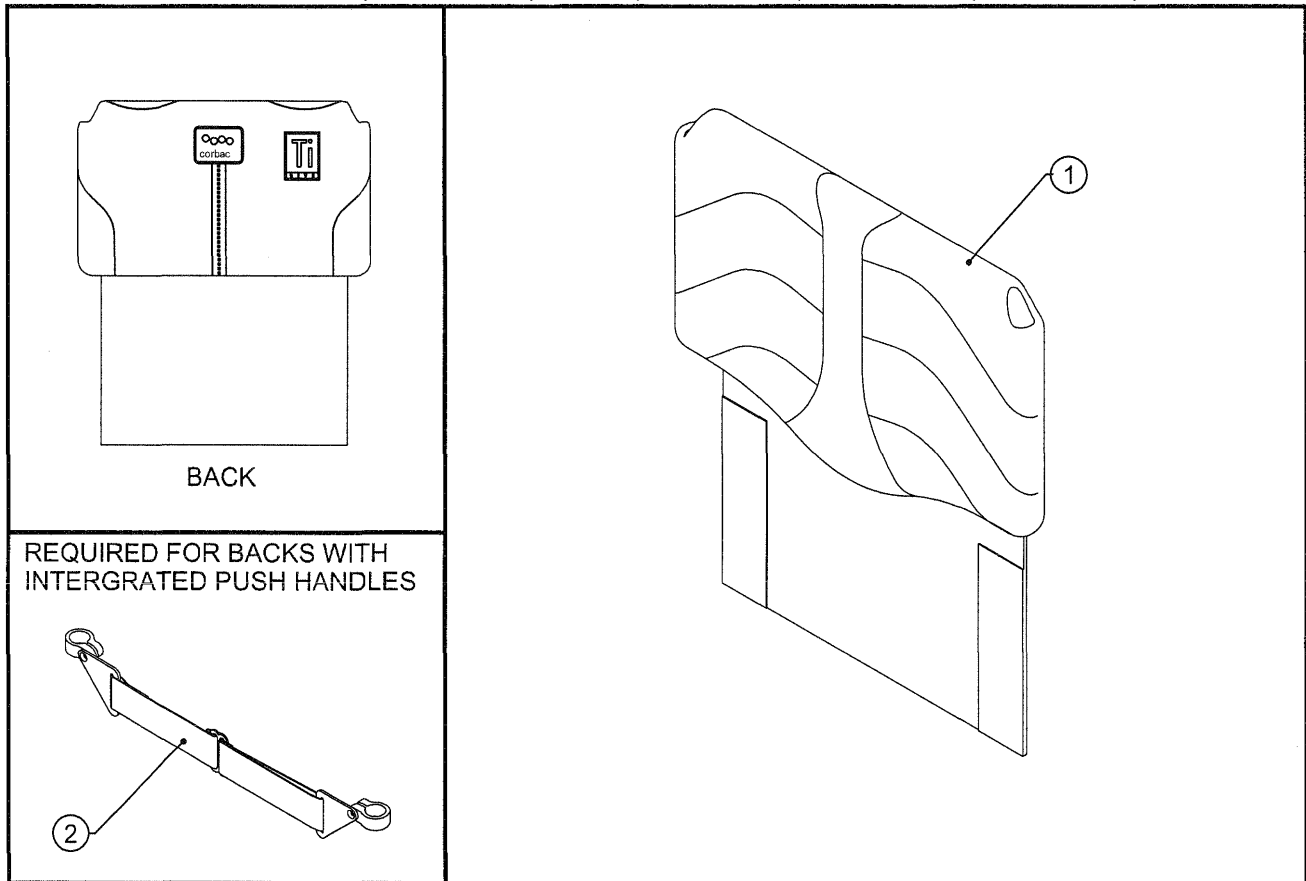

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103187	14" WIDE CORBAC
1B	1	CP103188	15" WIDE CORBAC
1C	1	CP103189	16" WIDE CORBAC
1D	1	CP103190	17" WIDE CORBAC
1E	1	CP103191	18" WIDE CORBAC
2	1	CP103192	CORBAC STRAP FOR PUSH HANDLES


TABLE ONE - AVAILABLE BACK HEIGHTS

CHAIR WIDTH	BACK HEIGHT W/O PUSH	BACK HEIGHT W/ PUSH
14"	12.5"- 15"	13"-15.5"
15"	13"- 16"	13"- 16"
16"	14"- 16"	14"- 16"
17"	14"- 16"	14"- 16"
18"	14"- 16"	14"- 16"

TiLite™ MEDIUM HEIGHT BACKREST WITH STANDARD EXTENSIONS
USED TO ACHIEVE AVAILABLE BACK HEIGHTS

CORBAC BACKREST UPHOLSTERY

TiLite™ X, SX, AERO X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

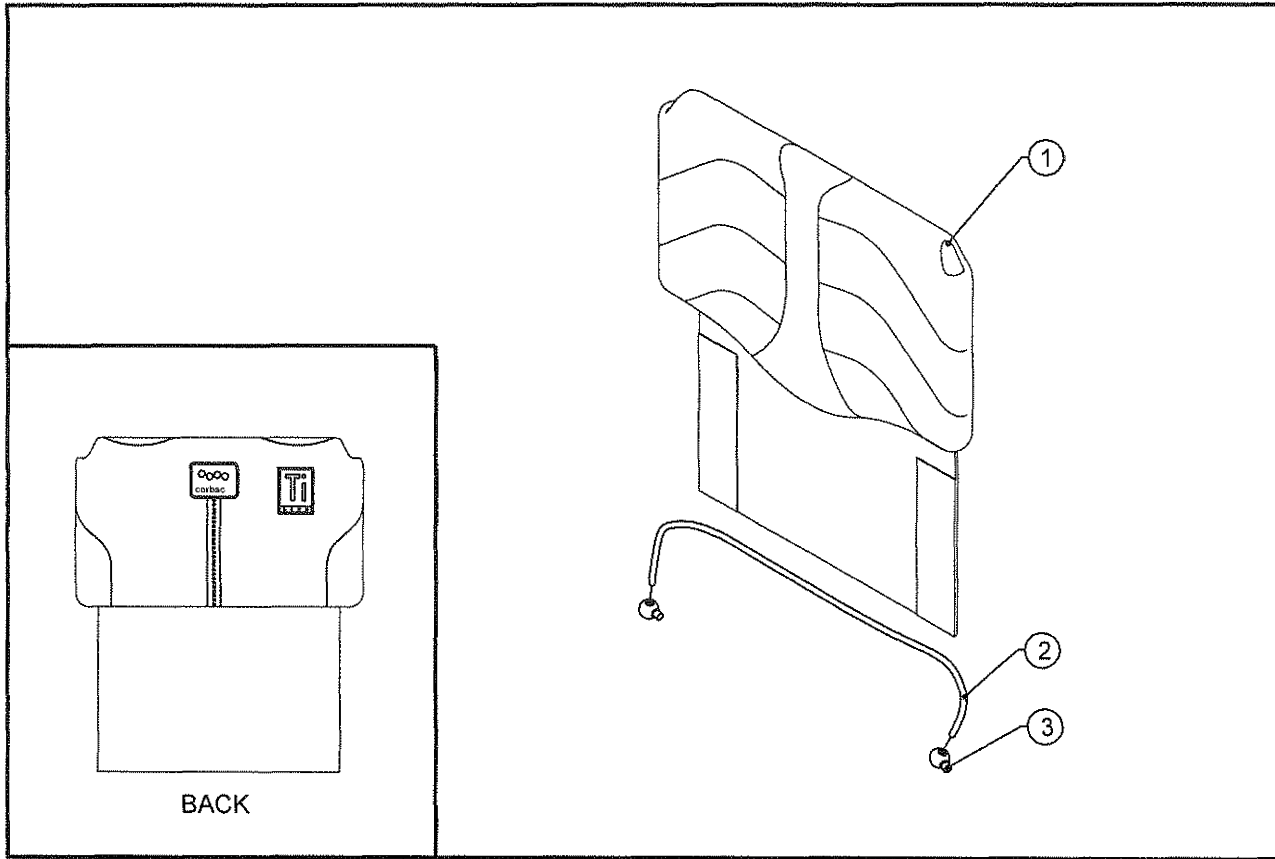

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	TABLE ONE	CORBAC BACKREST
2	1	RW100214	1/8" SHOCK CORD, BLACK
3	2	RW100026	BALL CORD LOCK, BLACK
1-3		TABLE ONE	CORBAC ASSY FOR X, SX, AERO X, 2GX

TABLE ONE - BACKREST ASSEMBLIES AND BACK STRAPS


CHAIR WIDTH	CORBAC BACKREST	X, SX, AERO X, 2GX BACK ASSY
14"	CP103187	AS111133
15"	CP103188	AS111134
16"	CP103189	AS111135
17"	CP103190	AS111136
18"	CP103191	AS111137

TABLE TWO - AVAILABLE BACK HEIGHTS

CHAIR TYPE	BACK HEIGHT W/O PUSH	BACK HEIGHT W/ PUSH	BACK HEIGHT W/ FOLDING PUSH
X, SX	11"-14"	11"-14"	11"-14"
AERO X, 2GX	11"-15"	11"-15"	11"-15"

FOAM SEAT CUSHIONS

ALL CHAIRS

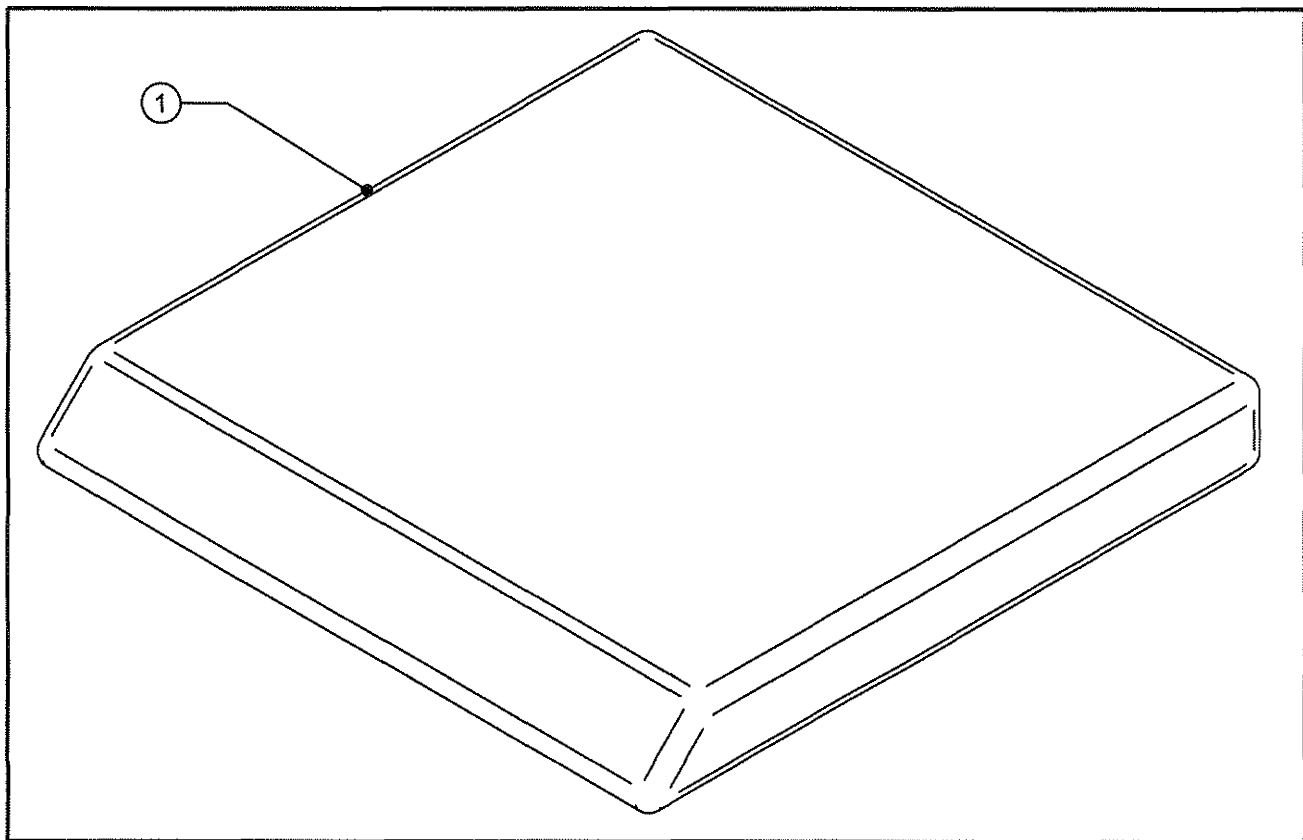

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE AND TWO	FOAM SEAT CUSHION ALL CHAIRS WHEN ORDERING PLEASE SPECIFY CUSHION THICKNESS 2", 3", OR 4"

TABLE ONE

WIDTH	DEPTH					
	8"	9"	10"	11"	12"	13"
8"	CP105488	CP105489	CP105490	CP105944	CP105945	CP106831
9"	CP105491	CP105492	CP105493	CP105494	CP105946	CP105947
10"	CP106813	CP105495	CP105496	CP105497	CP105498	CP105948
11"	CP106814	CP106818	CP105499	CP105500	CP105501	CP105502
12"	CP106815	CP106819	CP106822	CP105503	CP100054	CP100055
13"	CP106816	CP106820	CP106823	CP106826	CP100057	CP100058
14"	CP106817	CP106821	CP106824	CP106827	CP106830	CP100062

TABLE TWO


WIDTH	DEPTH						
	14"	15"	16"	17"	18"	19"	20"
10"	CP105949						
11"	CP105950	CP105951					
12"	CP100056	CP105952	CP105953	CP106844	CP106847	CP106851	CP106855
13"	CP100059	CP100060	CP100061	CP106845	CP106848	CP106852	CP106856
14"	CP100063	CP100064	CP100065	CP106846	CP106849	CP106853	CP106857
15"	CP100066	CP100067	CP100068	CP100069	CP106850	CP106854	CP106858
16"	CP106834	CP100070	CP100071	CP100072	CP100073	CP101611	CP101612
17"	CP106835	CP100074	CP100075	CP100076	CP100077	CP101613	CP101614
18"	CP106836	CP106840	CP100078	CP100079	CP100080	CP101615	CP101616
19"	CP106837	CP106841	CP101617	CP101618	CP101619	CP101620	CP101621
20"	CP106838	CP106842	CP101622	CP101623	CP101624	CP101625	CP101626

*CUSTOM - CP100798

* = CALL FACTORY TO SPECIFY THE SIZE OF THE CUSHION REQUIRED

HARD FOAM SEAT CUSHION

TiSport BB

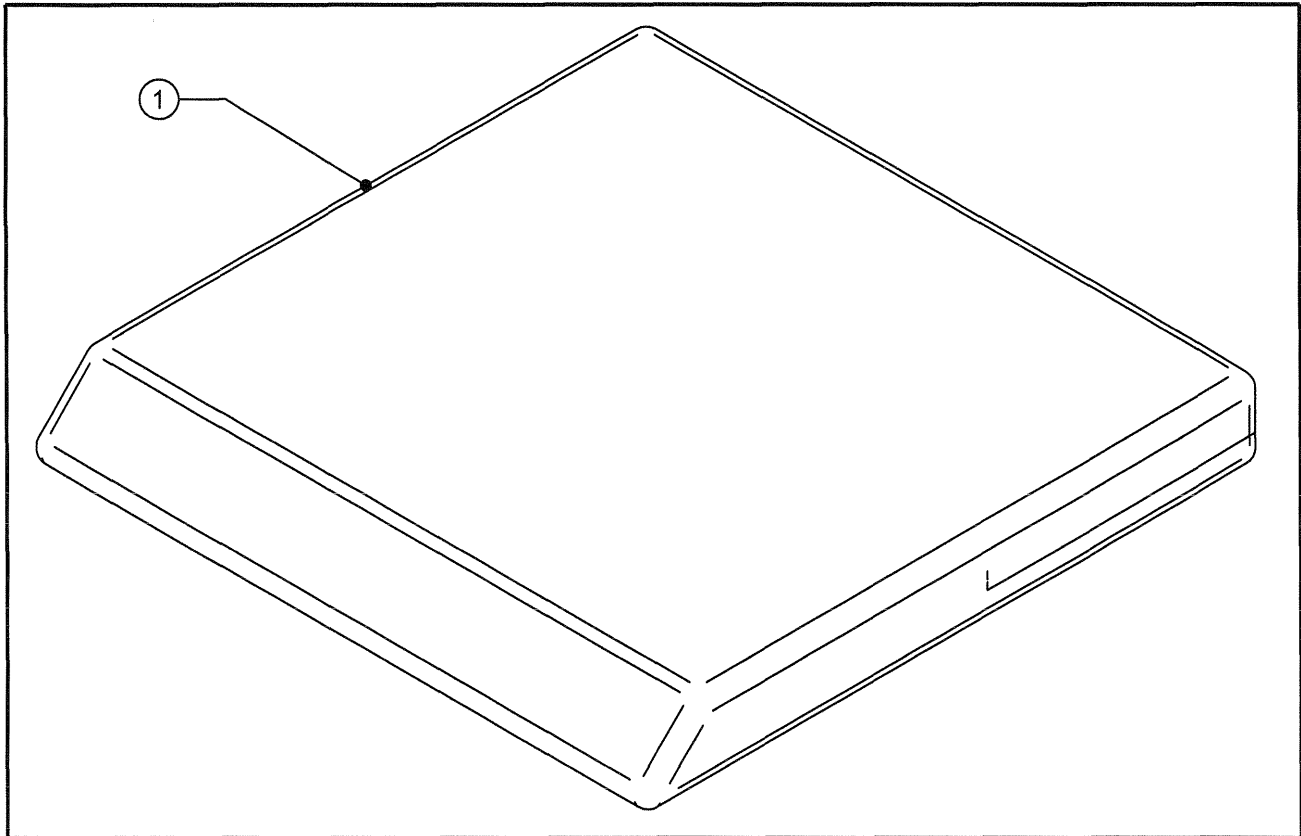


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP101675	HARD FOAM SEAT CUSHION WHEN ORDERING PLEASE SPECIFY CUSHION THICKNESS 2", 3", OR 4"

* = CALL FACTORY TO SPECIFY THE SIZE OF THE CUSHION REQUIRED



StimuLITE® SEAT CUSHION

by Supracor

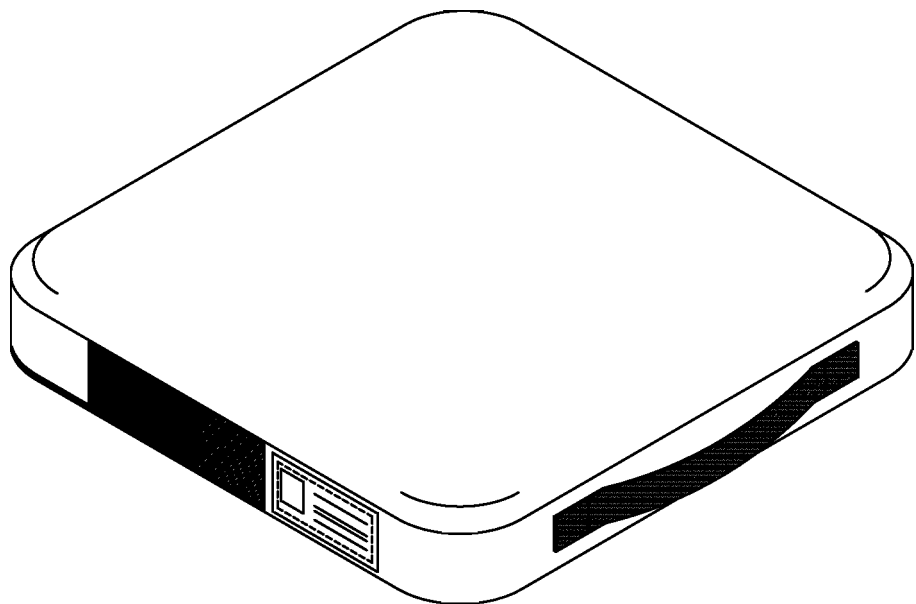


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100794	Supracor StimuLITE® SEAT CUSHION*

* = CONSULT FACTORY FOR SIZE WHEN ORDERING.
PLEASE NOTE THAT FOR A SUPERCORE CUSHION THE CHAIR ORDER
NUMBER OR SERIAL NUMBER OF THE CHAIR WILL BE REQUIRED




ROHO® SEAT CUSHIONS

by ROHO®

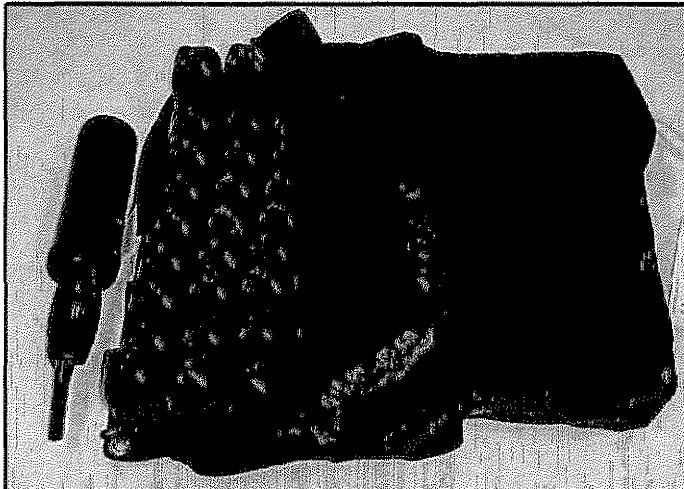

CUSTOM

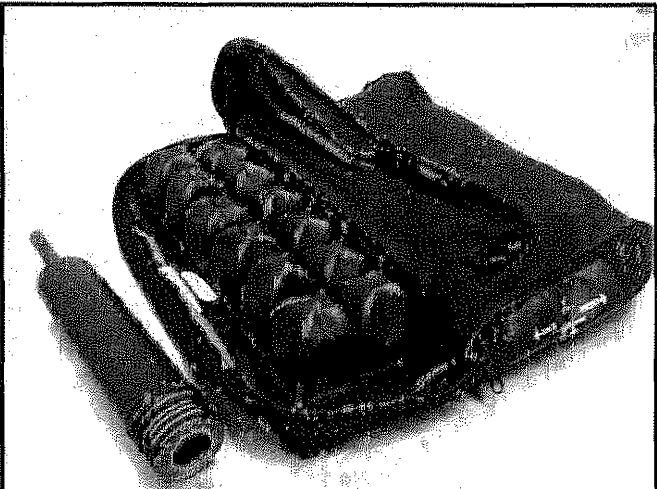

LOW PROFILE QUADTRO SELECT

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP101437	ROHO CUSTOM CUSHION - SPECIAL ORDER
1B	1	CP106903	ROHO CUSTOM CUSHION COVER - SPECIAL ORDER
2A	1	CP103103	ROHO LOW PROFILE QUADTRO SELECT CUSHION - SPECIAL ORDER
2B	1	CP103104	ROHO LOW PROFILE QUADTRO SELECT CUSHION COVER - SPECIAL ORDER

* = CONSULT FACTORY FOR SIZE WHEN ORDERING.
PLEASE NOTE THAT FOR A ROHO CUSHION THE CHAIR ORDER
NUMBER OR SERIAL NUMBER OF THE CHAIR WILL BE REQUIRED

VARILITE® SEAT CUSHIONS

by VARILITE®



ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103105	VARILITE EVOLUTION CUSHION - SPECIAL ORDER
1B	1	CP103106	VARILITE EVOLUTION CUSHION COVER - SPECIAL ORDER
2A	1	CP103107	VARILITE ZOID CUSHION - SPECIAL ORDER
2B	1	CP103108	VARILITE ZOID CUSHION COVER- SPECIAL ORDER

* = CONSULT FACTORY FOR SIZE WHEN ORDERING.
PLEASE NOTE THAT FOR A VARILITE CUSHION THE CHAIR ORDER
NUMBER OR SERIAL NUMBER OF THE CHAIR WILL BE REQUIRED