ROHO® DRY FLOATATION® Cushions
Operation Manual

Single Compartment, Dual Compartment, Sensor Ready

permobil

s h a p e f i t t i n g t e c h n o l o g y®

Contact Information

 **ROHO, Inc.**
 1501 South 74th St.
 Belleville, IL 62223 U.S.A.
 U.S.A.:
 800-736-0925
 Fax 888-551-3449
 Customer Support:
 orders.roho@permobil.com
 Outside the U.S.A.:
 618-277-9150
 Fax 618-277-6518
 intlorders.roho@permobil.com
 permobil.com

Warning

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

Caution

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

Incident Reporting

In the event of a serious incident or injury, contact Customer Support.

Contents

Intended Use	2
Important Safety Information	3-4
Parts Detail	5
Product Specifications	
Cushion Adjustment	6-7
Cleaning and Disinfecting	8-9
Troubleshooting	10
Storage, Transport, Disposal and Recycling	
Limited Warranty	
Español	11
Français	21
Português	31

Symbols Glossary

	Manufacturer		Disinfecting instructions
	Date of manufacture		Cleaning instructions
	Meets flammability		Front of product
	Weight limit		Back of product
	Medical Device		Bottom of product
	Serial number		Size (width x depth x height)
	Fits		Consult instructions for use
	Batch code		Model number for customer reference only
	Quantity		Manufacturer's catalog number (in UDI - unique device identification)

Please Note

Supplier: This manual must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Products may be covered by one or more U.S. and foreign patents and trademarks, including DRY FLOATATION®, HIGH PROFILE®, LOW PROFILE®, MID PROFILE®, ROHO®, Sensor Ready®, shape fitting technology®, Smart Check®, and Yellow Rope®. Zytel® is a registered trademark of DuPont.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

ROHO, Inc. is part of Permobil.

© 2020, 2022 Permobil

Rev. 2022-07-25

The following products are featured in this operation manual:

ROHO LOW PROFILE Dual Compartment Cushion

ROHO MID PROFILE Dual Compartment Cushion

ROHO HIGH PROFILE Dual Compartment Cushion

ROHO MID PROFILE Single Compartment Cushion

ROHO MID PROFILE Single Compartment Cushion with Sensor Ready Technology

Intended Use

Each ROHO DRY FLOATATION product in this manual is an adjustable, air-filled, cellular-design wheelchair support surface. There is no weight limit, yet the cushion must be properly sized to the individual.

Each of the following products utilizes DRY FLOATATION Technology and is intended to conform to an individual's seated shape to provide skin/soft tissue protection, positioning, and an environment to facilitate wound healing.

- ROHO LOW PROFILE Dual Compartment Cushion (LOW PROFILE Dual Compartment Cushion)
- ROHO MID PROFILE Dual Compartment Cushion (MID PROFILE Dual Compartment Cushion)
- ROHO HIGH PROFILE Dual Compartment Cushion (HIGH PROFILE Dual Compartment Cushion)

Each of the following products utilizes DRY FLOATATION Technology and is intended to conform to an individual's seated shape to provide skin/soft tissue protection and an environment to facilitate wound healing.

- ROHO MID PROFILE Single Compartment Cushion (MID PROFILE Single Compartment Cushion)
- ROHO MID PROFILE Single Compartment Cushion with Sensor Ready Technology (MID PROFILE Sensor Ready Cushion)

The Sensor Ready Cushion is compatible with the Smart Check Cushion Monitoring Device.

Contraindication: Based on clinical, scientific or engineering evidence, these products may not be suitable for individuals who require accommodation of pelvic asymmetry greater than is specified below, per product:

1 in. (2.5cm) - LOW PROFILE Dual Compartment Cushion

2 in. (5cm) - MID PROFILE Single Compartment Cushion; MID PROFILE Dual Compartment Cushion; MID PROFILE Sensor Ready Cushion

3 in. (7.5cm) - HIGH PROFILE Dual Compartment Cushion

A clinician who is experienced in seating and positioning should determine whether the product is appropriate for the individual's particular seating needs. The product's intended use statement alone should not be relied upon to make that determination.

The medical products featured in this document are intended to be only one part of an overall care regimen that includes all seating and mobility equipment and therapeutic interventions. A clinician who is experienced in seating and positioning should identify this care regimen after evaluating the individual's physical needs and overall medical condition.

A clinician should also evaluate for visual, reading, and cognitive impairment to determine the need for caregiver assistance or the need for other assistive technology, such as large-print instructions, to ensure proper product use.

Important Safety Information

Warnings:

- Skin/soft tissue breakdown can occur due to a number of factors, which vary by individual. Check skin frequently, at least once a day. Redness, bruising, or darker areas (when compared to normal skin) may indicate superficial or deep tissue injury and should be addressed. If there is any discoloration to skin/soft tissue, STOP USE immediately. If the discoloration does not disappear within 30 minutes after disuse, immediately consult a healthcare professional.
- If you are unable to perform any tasks described in this manual, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.
- DO NOT use the cushion as a water flotation device (e.g., a Life Preserver). It will NOT support you in water.
- The cushion surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the cushion will contact unprotected skin.
- DO NOT expose the cushion to high heat, open flames or hot ashes. Testing or certification claims, including for flammability, may no longer apply to this device when it is combined with other products or materials. Check testing and certification claims for all products used in combination with this device.
- Use caution when handling the cushion: Metal grommets on the corners may be a finger trap hazard.
- DO NOT place obstructions between the individual and the cushion. The cushion and the cover MUST be compatible sizes and MUST be used as directed in this manual. Except for the compatible covers listed in "Product Specifications" in this manual, placement of any items between the individual and the cushion: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.
- DO NOT use the cushion on top of, or in conjunction with, any other products or materials, except as indicated in this manual. Using non-compatible products or materials, whether inside or outside the cover: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.

Cautions:

- DO NOT use a pump, cover or repair kit other than those compatible with the product.
- DO NOT modify the cushion or any components. Doing so may result in damage to the product and will void the warranty.
- DO NOT allow the cushion to come into contact with oil-based lotions or lanolin. They may degrade the material.
- Prolonged exposure to ozone may degrade materials used in the cushion, affect the performance of the cushion, and void the product warranty.
- Changes in altitude may require adjustment to the cushion.
- Keep the cushion away from sharp objects.

Important Safety Information, continued

Cautions, continued:

- If the cushion has been in temperatures lower than 32° F (0° C) and exhibits unusual stiffness, allow the cushion to warm to room temperature.
- Periodically, check the cover and components for damage and replace if necessary.
- DO NOT use any of the following as a handle for carrying or pulling the cushion: inflation valve(s), hose(s), or quick disconnect (Sensor Ready Cushion). Carry the cushion by the cushion base or by the carrying handle on the Standard Cover. Using any other components to carry or pull the cushion may damage the product.

Warnings - Motor Vehicle Transportation:


- Failure to pay attention to these warnings could result in severe injury to the individual in the wheelchair or to others.
- Whenever possible, transfer out of the wheelchair and into a manufacturer-installed vehicle seat without the cushion, and use the vehicle's crash-tested occupant restraint system.
- The cushion must be correctly and securely installed, used, and maintained according to the instructions and warnings provided by the cushion manufacturer.
- When the wheelchair is used as the vehicle seat, the cushion should only be used with wheelchair frames that are equipped with securement points specified in ANSI/RESNA WC-4:2017, section 19 (WC19) or ISO 7176-19, and that have been successfully crash tested on a wheelchair in the forward-facing position.
- The wheelchair with the cushion must be used with an effective wheelchair securement system and a properly positioned, crash-tested pelvic and shoulder-belt restraint, or Wheelchair Tiedown and Occupant Restraint System (WTORS), following the manufacturer's instructions.
- If the cushion has been involved in an accident during transport, check the cushion for damage and replace if necessary. See the "Troubleshooting" section in this manual.
- For more information about safely transporting wheelchair users, refer to: <http://wctransportation-safety.umtri.umich.edu/ridesafe-brochure>.
- A ROHO QUADTRO SELECT Cushion was included in a wheelchair seating system that was dynamically tested by a third-party testing facility for use in a motor vehicle. The system tested meets all applicable criteria for wheelchair seating systems proposed in Section 5.1 of ANSI/RESNA WC-4:2017 Section 20 "Wheelchair Seating Systems for Use in Motor Vehicles."
- Test results apply to seating systems intended for use by adults and children with a total body mass greater than 23 kg (51 lb, or approximately 6 years of age) who are transported facing forward in their wheelchairs in all types of motor vehicles.
- To determine whether motor vehicle third-party testing has been performed with a specific seating system combination, refer to all provided equipment operation manuals or contact the equipment manufacturer.

Parts Detail

Package contents: cushion, cover, operation manual, product registration card, hand pump, repair kit, limited warranty supplement.

Instructions are provided with the cover.

To order replacement parts, contact the product provider or Customer Support.

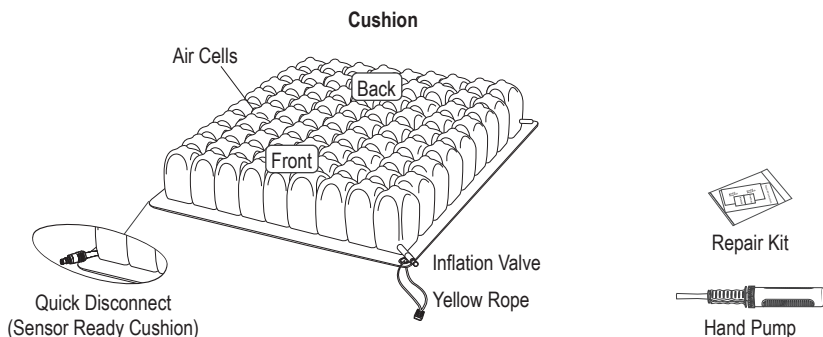The serial number label is located on the bottom of the cushion.

Product Specifications

Standard Cushion Dimensions and Weights

1 in. = 2.54 cm = 25.4 mm

Cushion:	SINGLE COMPARTMENT Cushions		DUAL COMPARTMENT Cushions		
	MID PROFILE (MP)	SENSOR READY MID PROFILE (MP)	LOW PROFILE	MID PROFILE	HIGH PROFILE
Height	3½ in. (8.5 cm)	3½ in. (8.5 cm)	2¼ in. (5.5 cm)	3¼ in. (8.5 cm)	4¼ in. (10.5 cm)
Average Weight*	3.0 lb. (1.4 kg)	3.0 lb. (1.4 kg)	2.5 lb. (1.1 kg)	3.0 lbs. (1.4 kg)	3.25 lbs. (1.5 kg)
Min. Width	7 cells; 13¼ in. / 33.5cm	7 cells; 13¼ in. / 33.5cm	6 cells; 11½ in. / 29cm	8 cells; 14¾ in. / 37.5cm	6 cells; 11½ in. / 29cm
Min. Depth	7 cells; 13¼ in. / 33.5cm	7 cells; 13¼ in. / 33.5cm	6 cells; 11½ in. / 29cm	9 cells; 16½ in. / 42cm	6 cells; 11½ in. / 29cm
Max. Width	11 cells; 20 in. / 50.5cm	13 cells; 23½ in. / 59.5cm	11 cells; 20 in. / 50.5cm	8 cells; 14¾ in. / 37.5cm	11 cells; 20 in. / 50.5cm
Max. Depth	11 cells; 20 in. / 50.5cm	12 cells; 21¼ in. / 55cm	11 cells; 20 in. / 50.5cm	9 cells; 16½ in. / 42cm	12 cells; 21¼ in. / 55cm

* based on 9 x 9 cell cushions: 16½ in. x 16½ in. (42 cm x 42 cm)

Note: Contact Customer Support for specific cushion sizes.

Cushion materials: cushion: neoprene rubber; made without latex; inflation valve: Zytel; Yellow Rope: nylon; quick disconnect: acetal.

Cover materials: Refer to the care label on the cover.

Cushion expected lifetime: 5 years

Covers and Accessories Compatible with ROHO DRY FLOATATION Cushions:

Not available in all markets. Contact Customer Support.

ROHO Heavy Duty Cushion Cover
ROHO Standard Cushion Cover
ROHO Planar Solid Seat Insert
ROHO Contour Base

IF YOU OWN A SMART CHECK:

The Sensor Ready Cushion is equipped with Sensor Ready Technology and can be used with Smart Check, which is a cushion feedback system. Smart Check can be used to find the individual's inflation level and help check their cushion. For more information, contact Customer Support.

NOTE: If you own a Smart Check, **STOP HERE** and refer to the Smart Check operation manual.

Cushion Adjustment

Warnings:


- Check inflation frequently, at least once a day!
- DO NOT use an under-inflated cushion or an over-inflated cushion because the product benefits will be reduced or eliminated, resulting in an increased risk to skin and other soft tissue. If the cushion does not appear to be holding air, or if you are not able to inflate or deflate the cushion, see "Troubleshooting". Immediately contact the clinical caregiver, distributor, or Customer Support if the problem persists.
- When possible, use hook and loop fasteners to further secure the cushion to the seating surface. Hook fasteners are available upon request.

Cover placement: Refer to the instructions provided with the cover.


Note: Keeping a cover on the cushion will help to protect the cushion components.

1. Inflate the cushion:


- a. Place the cushion in the wheelchair with the air cells facing up and the non-skid surface of the cover facing down.
- b. Slide the hand pump nozzle over the inflation valve. Open the inflation valve, at least one full rotation.
- c. Inflate until all of the air cells feel firm. Close the inflation valve until it comes to a stop. Remove the hand pump. Make sure the inflation valve is closed.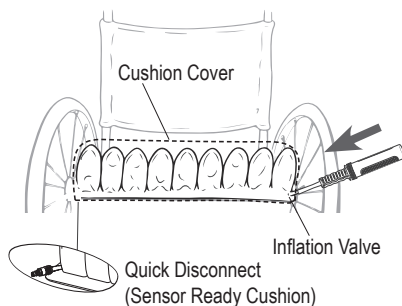

Note: See the next page.

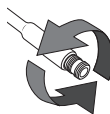Open

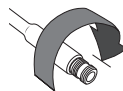
Close

Cushion Adjustment, continued

2. Adjust the cushion:

After overinflating the cushion, adjust to the Proper Cushion Inflation Setting:


- The individual should sit in the chair in their usual position. The cushion should be centered beneath the individual.
- Slide your hand between the cushion and the individual. Lift the leg slightly and feel for the lowest bony prominence, and then lower the leg to a sitting position.
- Open the inflation valve to remove air, keeping your hand beneath the individual's lowest bony prominence.
- Release air until you are barely able to move your fingertips. Close the valve. Make sure the inflation valve is closed.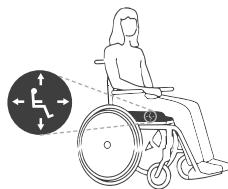

The seated figure shows product orientation.

Proper Cushion Inflation Setting

 = ½" - 1" (1.5 cm - 2.5 cm)

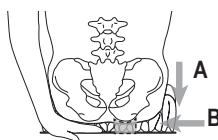A) "sink down" into the cushion and
B) "float" above the cushion base.

To adjust Dual Compartment Cushions: For individuals with a pelvic obliquity, first adjust the side with the deepest bony prominence. For front-to-back positioning, adjust the rear section first. Always re-check adjustment for each section.

Cleaning and Disinfecting

Note: For cover care instructions, refer to the manual provided with the cover.

Warnings:

- DO NOT allow water or cleaning solution to enter the cushion. Make sure the inflation valve(s) are closed. Sensor Ready Cushion: Make sure the Smart Check is detached from the cushion. Water in a cushion with Sensor Ready (SR) technology can result in damage to the Smart Check and cause a false reading when using Smart Check.
- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. Before use by a different individual: clean, disinfect, and check the product for proper functioning.
- Clean the product regularly and when it becomes soiled.
- Follow all manufacturer bleach container and germicidal disinfectant safety guidelines.
- Note about sterilization: High temperatures accelerate aging and will damage the product assembly. The products featured in this manual are not packaged in a sterile condition, nor are they intended or required to be sterilized prior to use. If institution protocol demands sterilization: First, follow the cleaning and disinfecting instructions; then, open the inflation valve and use the lowest possible sterilizing temperature, but not to exceed 70°C (158°F), for the shortest time possible. DO NOT autoclave.

Cautions:

- Use of the following may cause damage to the cushion: abrasives (steel wool, scouring pads); caustic, automatic dishwasher detergents; cleaning products that contain petroleum or organic solvents, including acetone, toluene, Methyl Ethyl Ketone (MEK), naphtha, dry cleaning fluid, adhesive remover; oil-based lotions, lanolin, ozone gas; ultraviolet light cleaning methods.
- DO NOT expose to direct sunlight.

Care Symbols

Machine wash hot, normal, at the temperature shown.

Machine wash gentle, warm.

Hand wash.

Line dry in the shade.

Use a handheld dryer.

Bleach (1 part bleach: 9 parts water)

Hand Wash and Hand Disinfect the Cushion (Preferred Method)

Hand Wash the Cushion: Remove the cover, close the inflation valve(s), and place in a large sink. Hand wash, using mild liquid hand soap, hand dishwashing liquid, laundry detergent, or a multipurpose detergent (Follow product label instructions.) Gently scrub all cushion surfaces, using a soft plastic bristle brush, a sponge or a cloth. Rinse with clean water. Air dry.

Hand Disinfect the Cushion: Follow the hand wash instructions, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. Rinse thoroughly with clean water. Air dry.

Caution: Rinse thoroughly. Detergent residue may cause air cells to stick together.

Cleaning and Disinfecting, continued

Machine Washing and Disinfecting the Cushion

 **IMPORTANT:** Machine washing can cause wear and damage, including cuts and tears to the cushion and the detachment of required product information. Washing and disinfecting the cushion by hand and air drying is strongly recommended.

 Warnings:

- DO NOT allow water or cleaning solution to enter the cushion. Make sure the inflation valve(s) are closed. Sensor Ready Cushion: Make sure the Smart Check is detached from the cushion. Water in a cushion with Sensor Ready (SR) technology can result in damage to the Smart Check and cause a false reading when using Smart Check.
- High water temperature may cause cushion discoloration and label detachment.
- DO NOT expose cushions to temperatures above 70°C (158°F), which may damage cushion components.
- For machine washing or disinfecting, make sure the washing machine capacity is sufficient. A washing machine that isn't large enough may not remove all soil or contaminants from a cushion.
- Using a damaged cushion will reduce or eliminate the benefits of the cushion and increase risk to skin and other soft tissue.

Prepare the Cushion for Machine Cleaning:

- 1) Remove the cover and close the inflation valve. Examine the cushion for damage, and patch any holes (refer to Repair Kit instructions provided with the product).
- 2) Remove stains or odors as needed: Place the cushion in warm water and a disinfectant cleaning agent. Gently scrub, using a soft plastic bristle brush, and rinse with clean water.
- 3) Deflate the cushion: Open the inflation valve and roll up the cushion, starting at the corner opposite the valve. After you have removed as much air as possible, close the inflation valve.
- 4) Inspect for any sharp edges in the washing machine that may damage the cushion.

Machine Wash the Cushion:

Place the cushion in a laundry bag and then into the washing machine. When folding the cushion, make sure the air cells face outward. Use warm water (85° - 105°F / 30° - 40°C) and laundry detergent or machine- and cushion-safe cleaner/biocide, placed in the detergent dispenser. Gentle cycle: Use a machine setting that does not include a high-speed spin cycle.

Machine Disinfect the Cushion:

Follow the machine washing instructions and use hot water 60°C (140°F). Use biocide (refer to the manufacturer instructions). Rinse with clean water. Remove the cushion from the laundry bag and air dry. For faster drying, use a hand-held hair dryer, hold no closer than 6 in. (15 cm) from the cushion, and use a sweeping motion.

 **Caution:** Rinse thoroughly. Detergent residue may cause air cells to stick together.

 **Warning:** Overheating the neoprene rubber can cause permanent product damage.

Troubleshooting

For additional assistance, contact the equipment provider, distributor, or Customer Support.

Not Holding Air	Inflate the cushion. Inspect the quick disconnect (Sensor Ready Cushion), the inflation valve(s), and hose(s) for damage. Confirm that the inflation valve(s) are completely closed. Look for holes in the cushion. If very small holes or no holes are visible, follow the instructions in the Repair Kit provided with the product. For inflation valve, quick disconnect, or hose damage, or for large holes or leaks in the cushion, see the Limited Warranty supplement, or contact Customer Support.
Not Able To Inflate or Deflate The Cushion	Confirm that the inflation valve(s) are open. Inspect the quick disconnect (Sensor Ready Cushion), the inflation valve(s) and hose(s) for damage.
Uncomfortable or Unstable	Confirm that: - the cushion is not underinflated or overinflated. (See "Cushion Adjustment") - the cushion cells and the top of the cover are facing up. - the cushion is centered beneath the individual. - the cushion is properly sized to the individual and the wheelchair. If the cushion still feels uncomfortable or unstable after checking all of the above, immediately consult the prescribing healthcare professional.
The Cushion Slides On the Wheelchair	Make sure the non-skid bottom of the cushion cover is facing away from the individual. Use directional labels on the cushion and on the cover to orient the cushion. When possible, use hook and loop fasteners to further secure the cushion to the wheelchair. Hook fasteners are available upon request. Refer to the instructions provided with the cover.
Component Is Damaged.	Contact Customer Support for a replacement.

Storage, Transport, Disposal and Recycling

Storage or Transport: Clean and disinfect the product. Open the inflation valve(s). Deflate the product. Place in a container that protects the product from moisture, contaminants and damage. After storing or transporting, inspect the product to ensure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/ regional regulations. Incineration must be performed by a qualified, licensed waste management facility.

Recycling: Contact a local recycling agency to determine recycling options for the product.

Limited Warranty

Limited Warranty Term from the date the product was originally purchased: 24 months. The warranty does not apply to punctures, tears, burns, or misuse. See also the Limited Warranty supplement provided with the product, or contact Customer Support.

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