


MyPermobil—Activation guide

Note: Wi-Fi is required for set-up

Getting started

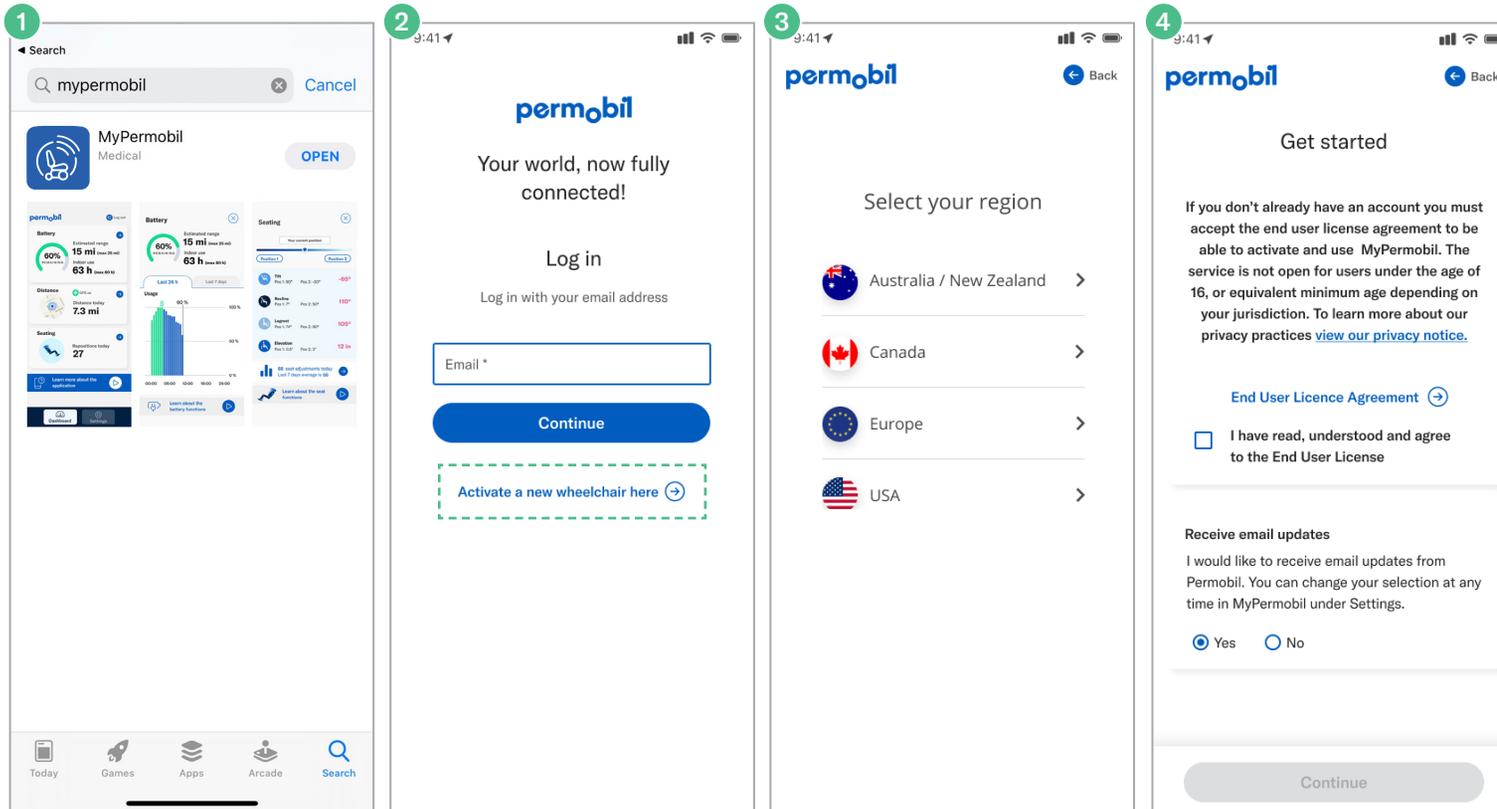

Confirm with provider that wheelchair was ordered with Permobil Connect

Click on activate a new wheelchair

Choose your region

Read/accept MyPermobil policy

Download MyPermobil on your Android or Apple phone

MyPermobil


Available on the App Store

MyPermobil connectivity is an available option on all new M and F-Series Corpus & Corpus VS models. Be sure to ask for Permobil Connect at time of order!


MyPermobil—Activation guide

Note: Wi-Fi is required for set-up


Create account

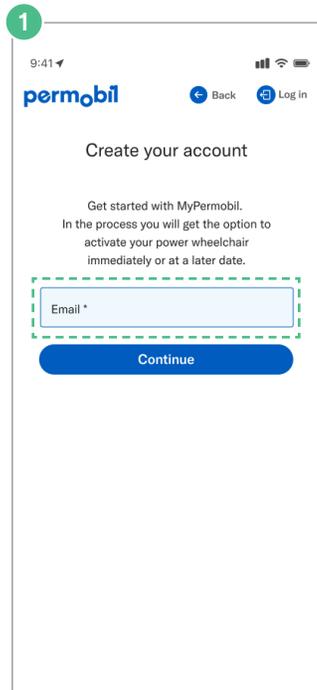

Enter email address

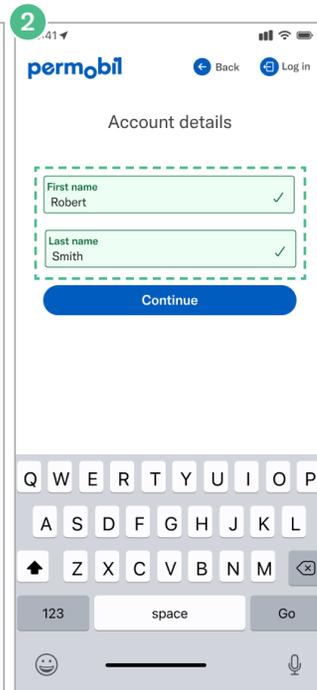


Enter your name

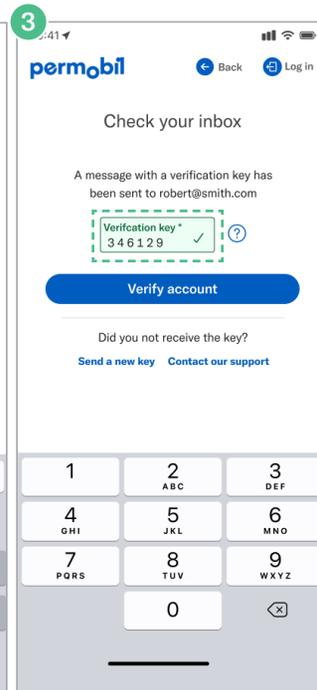


Check email, click “Verify” in email (if opened on same device as MyPermobil app) and follow prompts
- OR -
Enter verification code found in email, tap “Verify account”


Activate

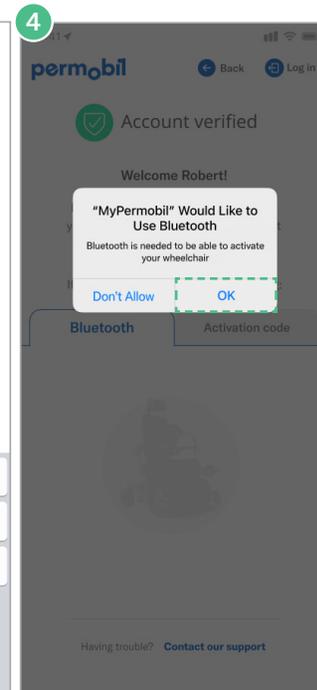

Accept to allow Bluetooth™, and turn on your wheelchair

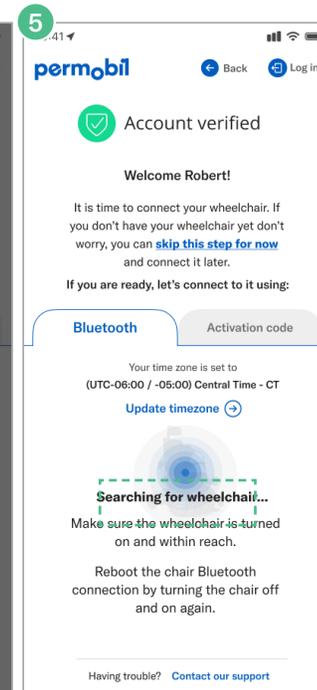

You may need to turn your wheelchair off and on again to reboot your chair’s Bluetooth connection
You may also activate via the activation code if there is difficulty with Bluetooth connection.

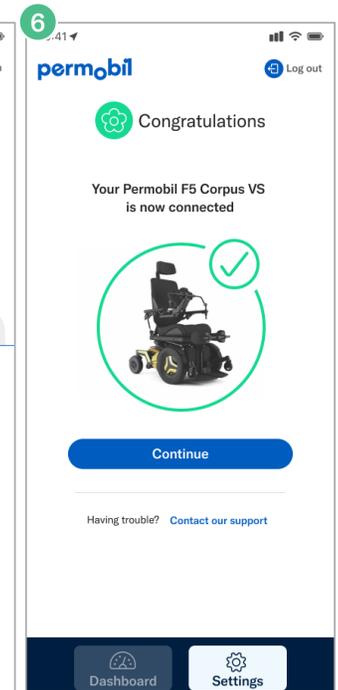

You are connected!