


Axle Plates and Axles

Section 6



ALUMINUM CAMBER TUBE

TiLite™ TR, TRC, YR, TRA, YRA, EVO, YR, ZR, ZRA, Aero Z, Aero R, TR (SERIES 2),
 YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2),
 TR (SERIES 3), AERO T, TRA (SERIES 2)

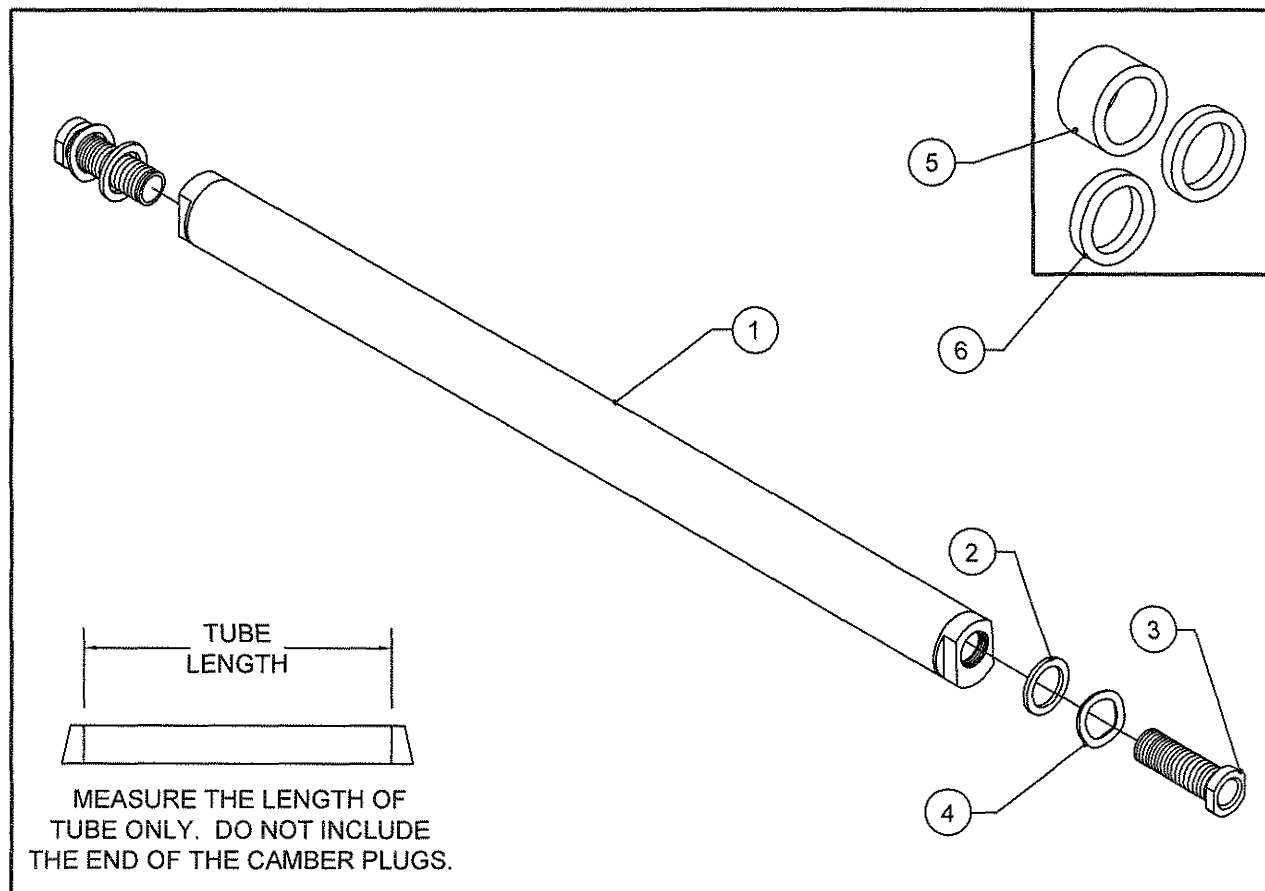

ITEM	QUANTITY	PART NO.	DESCRIPTION
CON. 1	1	TABLE ONE	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18° CAMBER TUBE
2	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .036 THICK
3	2	CP100151	AXLE SLEEVE
4	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
5	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
6	4	HD100089	WASHER .906" X .638" X .125
2-4		AS116180	AXLE SLEEVE PACKAGE
5-6		AS100945	REAR WHEEL SPACING PACKAGE
1-4		TABLE TWO	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18° CAMBER TUBE ASSY

TABLE ONE - FIXED CAMBER TUBE

TABLE TWO - FIXED CAMBER TUBE ASSEMBLY

CAMBER	PART NO.
0°	CP100419
2°	CP100420
4°	CP100421
6°	CP100999
8°	CP100422
12°	CP100423
15°	CP100424
18°	CP100425

CAMBER	ASSY NO.
0°	AS100029
2°	AS100058
4°	AS100030
6°	AS100604
8°	AS100031
12°	AS100032
15°	AS100078
18°	AS100079


CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING



Parts Manual

TITANIUM CAMBER TUBE

TiLite™ TR, TRC, YR, TRA, YRA, EVO, YR, ZR, ZRA, Aero Z, Aero R, Z10, TR (SERIES 2), YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3), AERO T, TRA (SERIES 2)

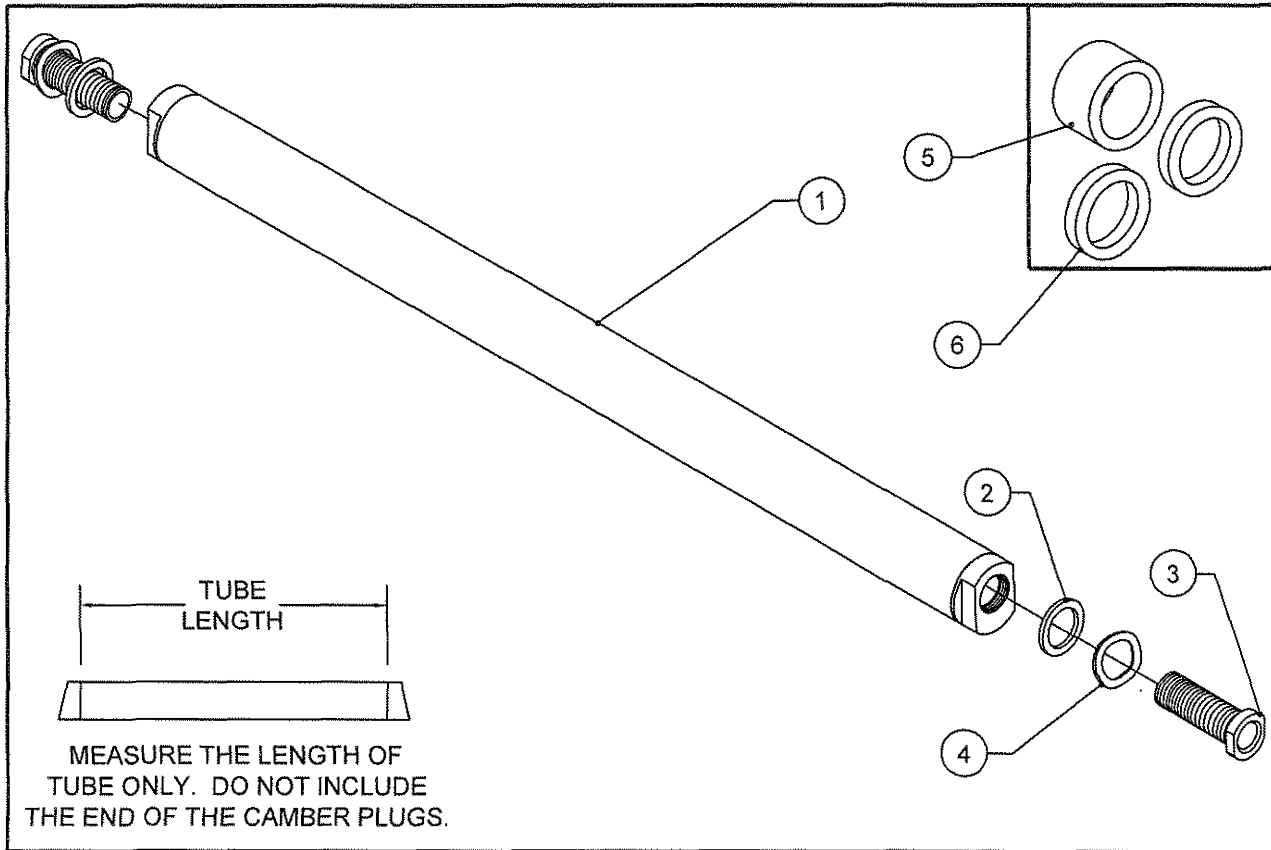


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18° TITANIUM CAMBER TUBE ASSY
2	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .036 THICK
3	2	CP100151	AXLE SLEEVE, QUICK RELEASE
4	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
5	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
6	4	HD100089	WASHER .906" X .638" X .125
2-4		AS116180	AXLE SLEEVE PACKAGE
5-6		AS100945	REAR WHEEL SPACING PACKAGE
1-4		TABLE ONE	TITANIUM CAMBER TUBE ASSY

TABLE ONE - TITANIUM CAMBER TUBE ASSY

° CAMBER	ASSY NO.
0°	AS102895
2°	AS102896
4°	AS102897
6°	AS102898
8°	AS102899
12°	AS102900
15°	AS102901
18°	AS102902

CARBON FIBER CAMBER TUBE

TiLite™ Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2),
TR (SERIES 3), AERO T, TRA (SERIES 2)

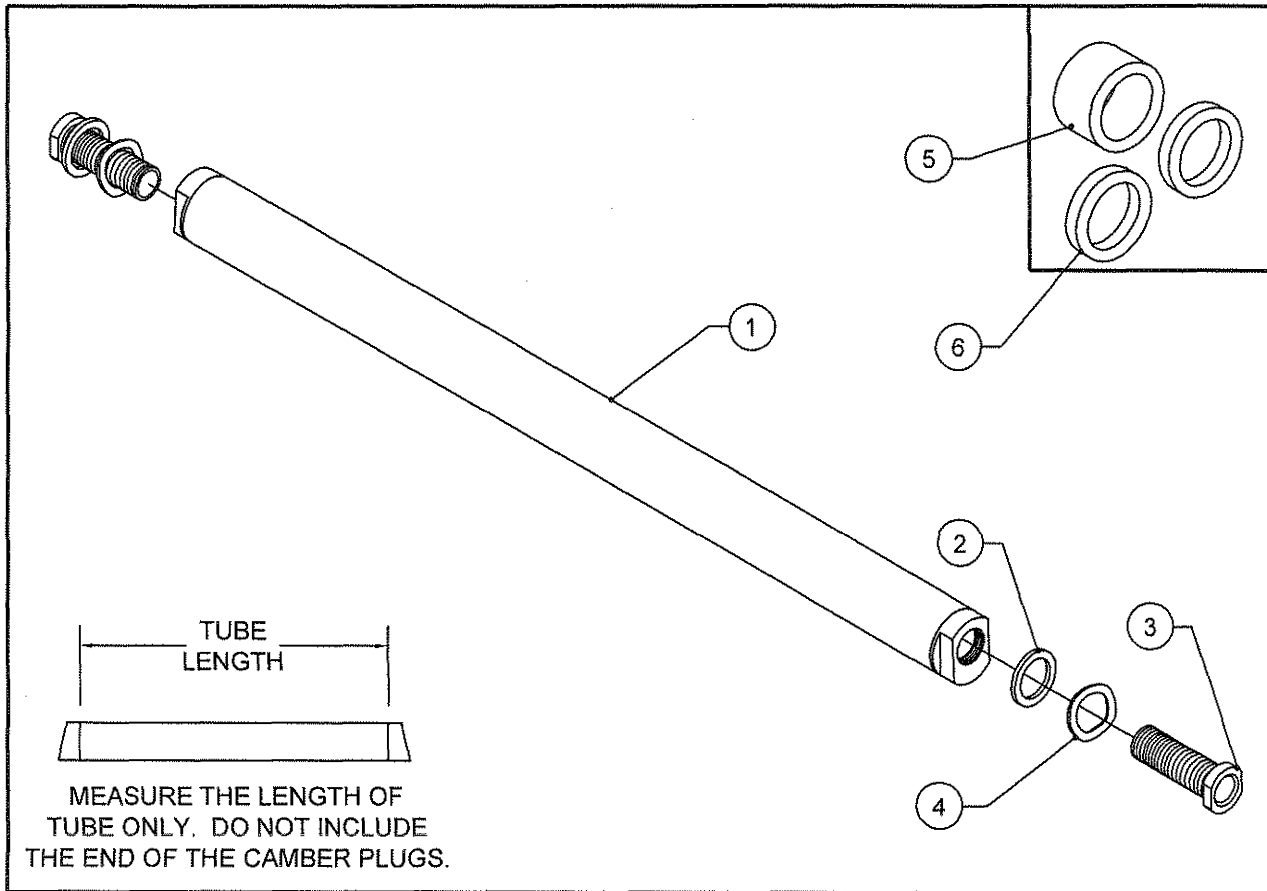

ITEM	QUANTITY	PART NO.	DESCRIPTION
CON. 1	1	TABLE ONE	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18° CARBON FIBER CAMBER TUBE
2	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .036 THICK
3	2	CP100151	AXLE SLEEVE
4	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
5	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
6	4	HD100089	WASHER .906" X .638" X .125
2-4		AS116180	AXLE SLEEVE PACKAGE
5-6		AS100945	REAR WHEEL SPACING PACKAGE
1-4		TABLE TWO	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18° CARBON FIBER CAMBER TUBE ASSY

TABLE ONE - FIXED CAMBER TUBE

CAMBER	PART NO.
0°	CP103845
2°	CP103846
4°	CP103847
6°	CP103848
8°	CP103849
12°	CP103850
15°	CP103851
18°	CP103852


TABLE TWO - FIXED CAMBER TUBE ASSEMBLY

CAMBER	ASSY NO.
0°	AS104327
2°	AS104328
4°	AS104329
6°	AS104330
8°	AS104331
12°	AS104332
15°	AS104333
18°	AS104334

CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING

ALUMINUM ZERO PLAY CAMBER TUBE

TiLite™ TR, TRC, YR, TRA, YRA, EVO, YR, ZR, ZRA, AERO Z, AERO R, TR (SERIES 2),
YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3),
AERO T, TRA (SERIES 2)

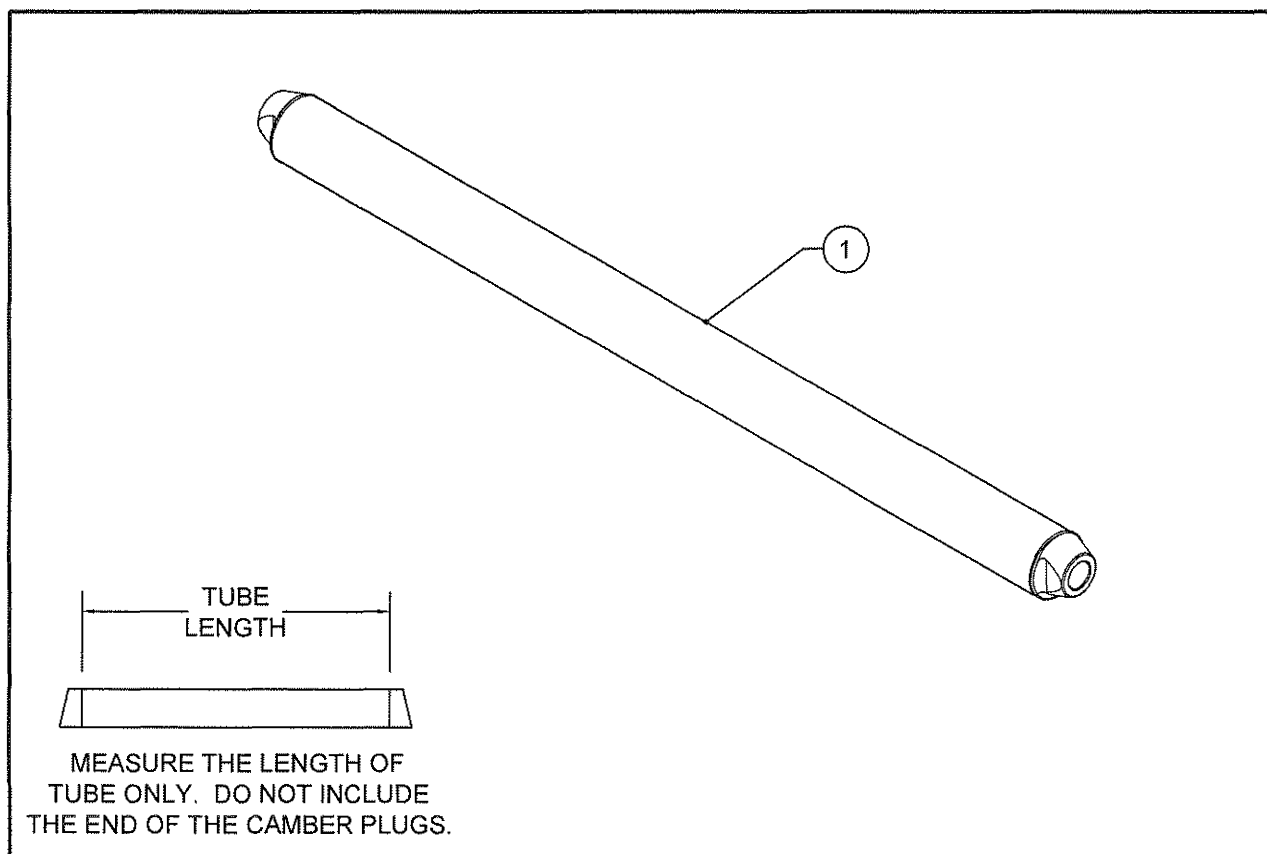

ITEM	QUANTITY	PART NO.	DESCRIPTION
CON. 1	1	TABLE ONE	0°, 2°, 4°, 6° CAMBER TUBE


TABLE ONE - FIXED CAMBER TUBE

CAMBER	PART NO.	ASSY NO.
0°	CP107231	AS115958
2°	CP107232	AS115959
4°	CP107233	AS115960
6°	CP107234	AS115961

CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING

TITANIUM ZERO PLAY CAMBER TUBE

TiLite™ TR, TRC, YR, TRA, YRA, EVO, YR, ZR, ZRA, AERO Z, AERO R, Z10, TR (SERIES 2),
YR (SERIES 2), ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2), TR (SERIES 3),
AERO T, TRA (SERIES 2)

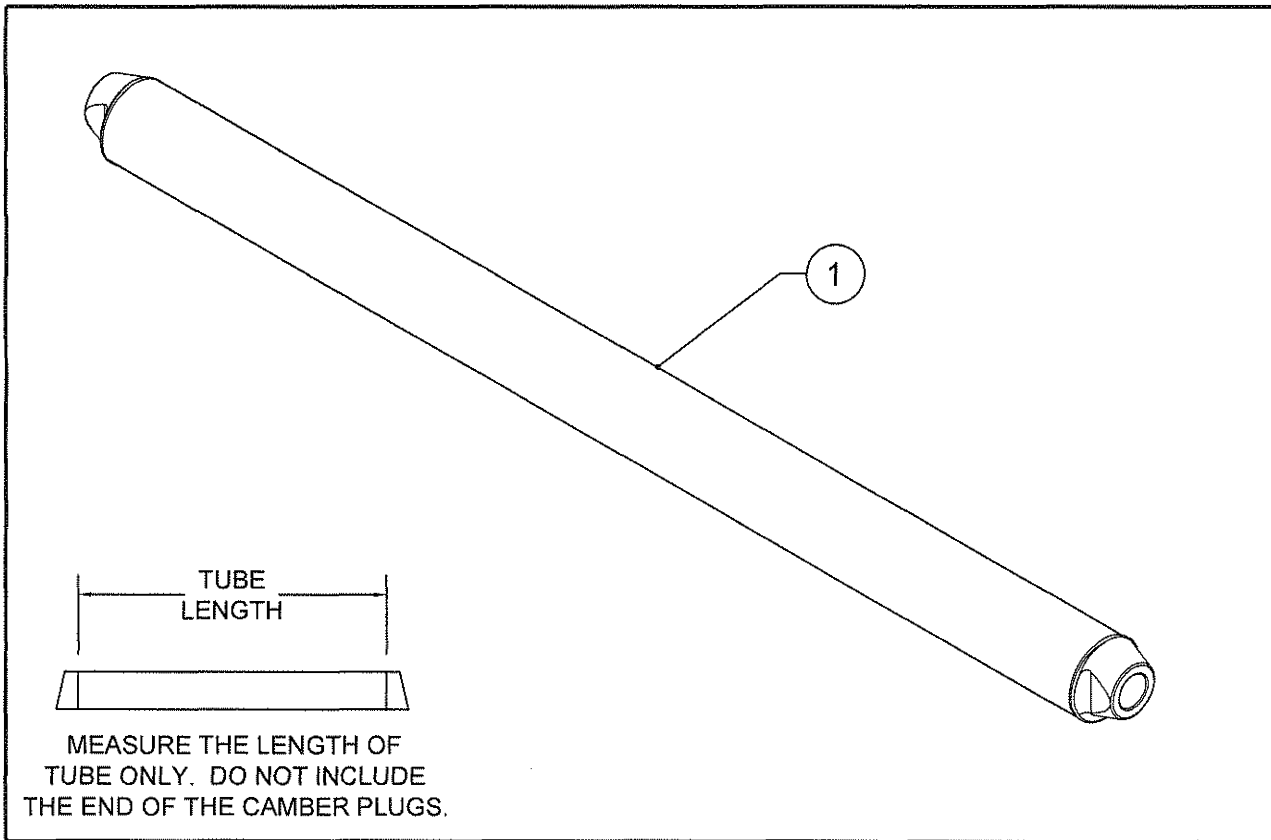


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	TABLE ONE	0°, 2°, 4°, 6° TITANIUM ZERO PLAY CAMBER TUBE ASSY

TABLE ONE - TITANIUM ZERO PLAY CAMBER TUBE ASSY

° CAMBER	PART NUMBER	ASSY NO.
0°	CP107235	AS115962
2°	CP107236	AS115963
4°	CP107237	AS115964
6°	CP107238	AS115965

CARBON FIBER ZERO PLAY CAMBER TUBE

TiLite™ Z10, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2),
TR (SERIES 3), AERO T, TRA (SERIES 2)

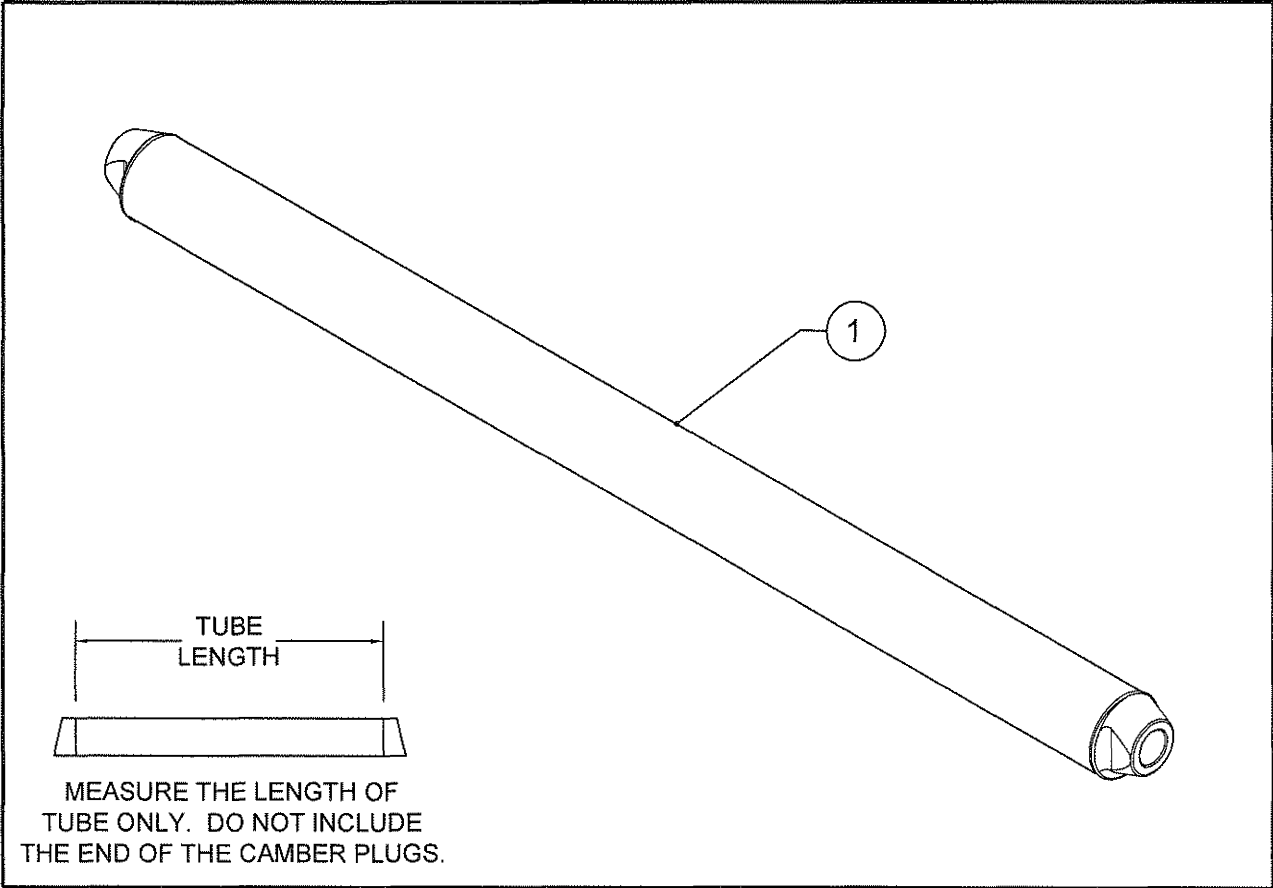

ITEM	QUANTITY	PART NO.	DESCRIPTION
CON. 1	1	TABLE ONE	0°, 2°, 4°, 6° CARBON FIBER ZERO PLAY CAMBER TUBE


TABLE ONE - FIXED CAMBER TUBE

CAMBER	PART NO.	ASSY NO.
0°	CP107239	AS115966
2°	CP107240	AS115967
4°	CP107241	AS115968
6°	CP107242	AS115969

CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING

EXPANDABLE CAMBER TUBE ASSY

TiLite™ YG, YGS

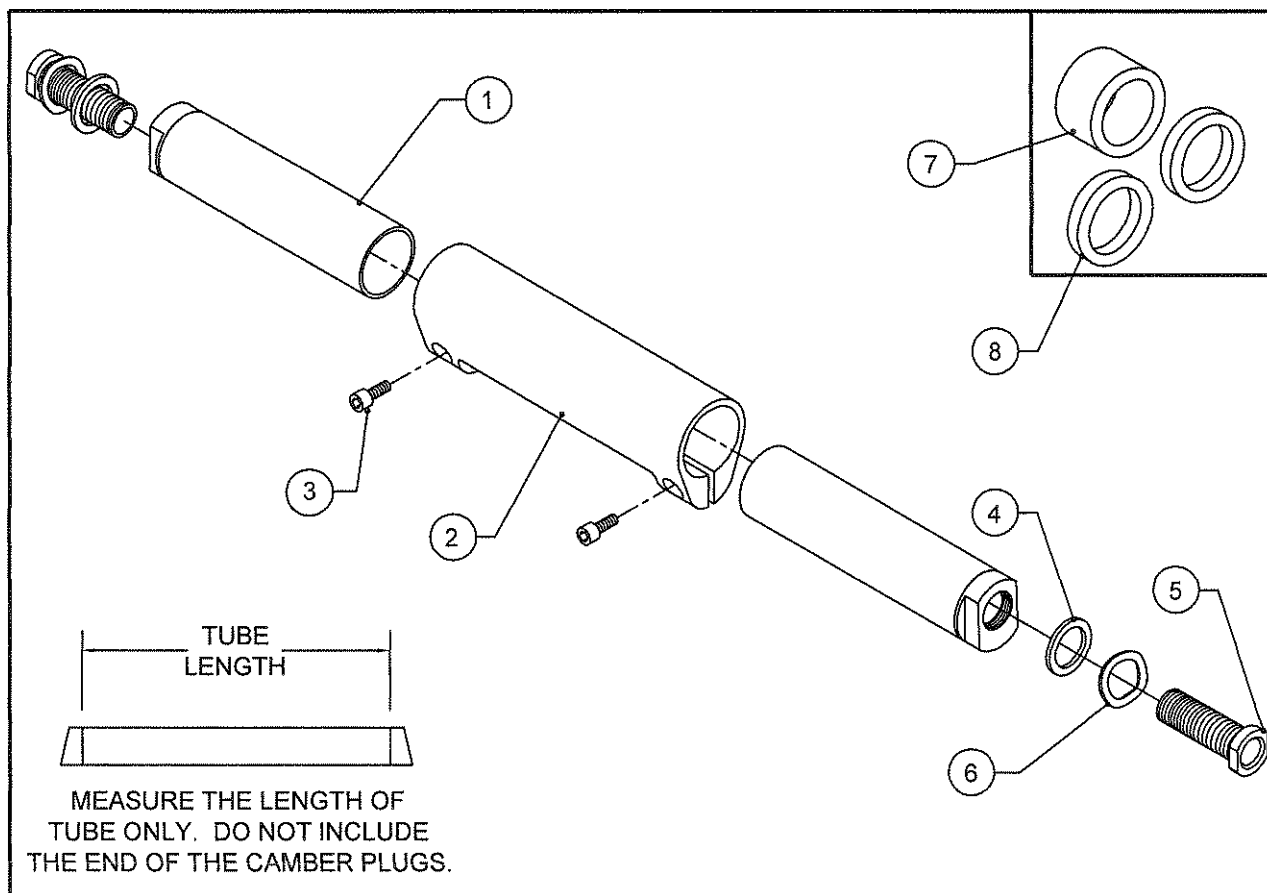

ITEM	QUANTITY	PART NO.	DESCRIPTION
CON. 1	1	TABLE ONE	0°, 2°, 4°, 6°, 8°, 12° CAMBER TUBE
2	2	CP101943	CAMBERTUBE EXPANSION BRACKET
3	2	HD100050	1/4-20 X 3/4" SOCKET HEAD CAP SCREW
4	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .036 THK
5	2	CP100151	AXLE SLEEVE
6	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
7	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
8	4	HD100089	WASHER .906" X .638" X .125
4-6		AS116180	AXLE SLEEVE PACKAGE
7-8		AS100945	REAR WHEEL SPACING PACKAGE
1-6		TABLE TWO	0°, 2°, 4°, 6°, 8°, 12° CAMBER TUBE ASSY

TABLE ONE

° CAMBER	ASSY NO.
0°	CP101911
2°	CP101912
4°	CP101913
6°	CP101914
8°	CP101915
12°	CP101916


TABLE TWO

° CAMBER	ASSY NO.
0°	AS102021
2°	AS102022
4°	AS102023
6°	AS102024
8°	AS102025
12°	AS102026

CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING

CAMBER TUBE

TiLite™ TWIST

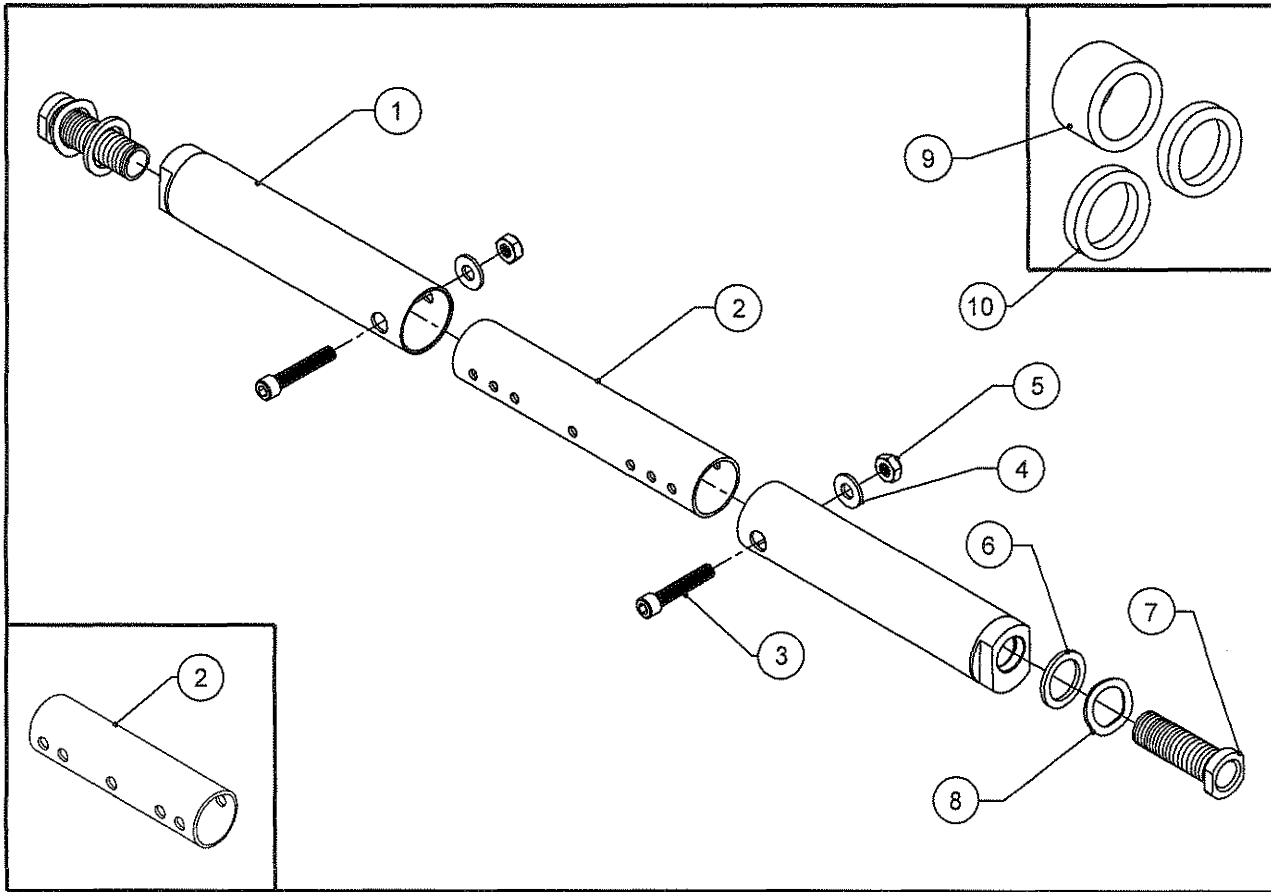

ITEM	QUANTITY	PART NO.	DESCRIPTION
CON. 1	2	TABLE ONE	PED CAMBER TUBE
2A	1	CP105279	PED CAMBER TUBE SPACER 8"
2B	1	CP105160	PED CAMBER TUBE SPACER
3	2	HD100678	1/4-28 X 1-1/2" SHCS, BLACK ZINC
4	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
5	2	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
6	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .036 THICK
7	2	CP100151	AXLE SLEEVE
8	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
9	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
10	4	HD100089	WASHER .906" X .638" X .125
6-8		AS116180	AXLE SLEEVE PACKAGE
9-10		AS100945	REAR WHEEL SPACING PACKAGE
CON. 1,2A,-8		TABLE ONE	0°, 2°, 4°, 6°, 8°, 12°, 15° CAMBER TUBE ASSY FOR 8" WIDTH
CON. 1,2B,-8		TABLE ONE	0°, 2°, 4°, 6°, 8°, 12°, 15° CAMBER TUBE ASSY FOR 9"-15" WIDTH


TABLE ONE

CAMBER	8" WIDTH CAMBER TUBE PART NO.	9"-15" WIDTH CAMBER TUBE PART NO.	8" WIDTH ASSY NO.	9"-15" WIDTH ASSY NO.
0°	AS111781	AS111783	AS111787	AS111791
2°	AS111782	AS111784	AS111788	AS111792
4°	AS111785	AS111785	AS111789	AS111793
6°	AS111786	AS111786	AS111790	AS111794
8°	AS111986	AS111986	AS111989	AS111989
12°	AS111987	AS111987	AS111990	AS111990
15°	AS111988	AS111988	AS111991	AS111991

CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING

AXLE ASSY

ADJUSTABLE CENTER OF GRAVITY, TiLite™ TR, TRC, BB, YR
TR(SERIES 2), TRC(SERIES 2), YR(SERIES 2)

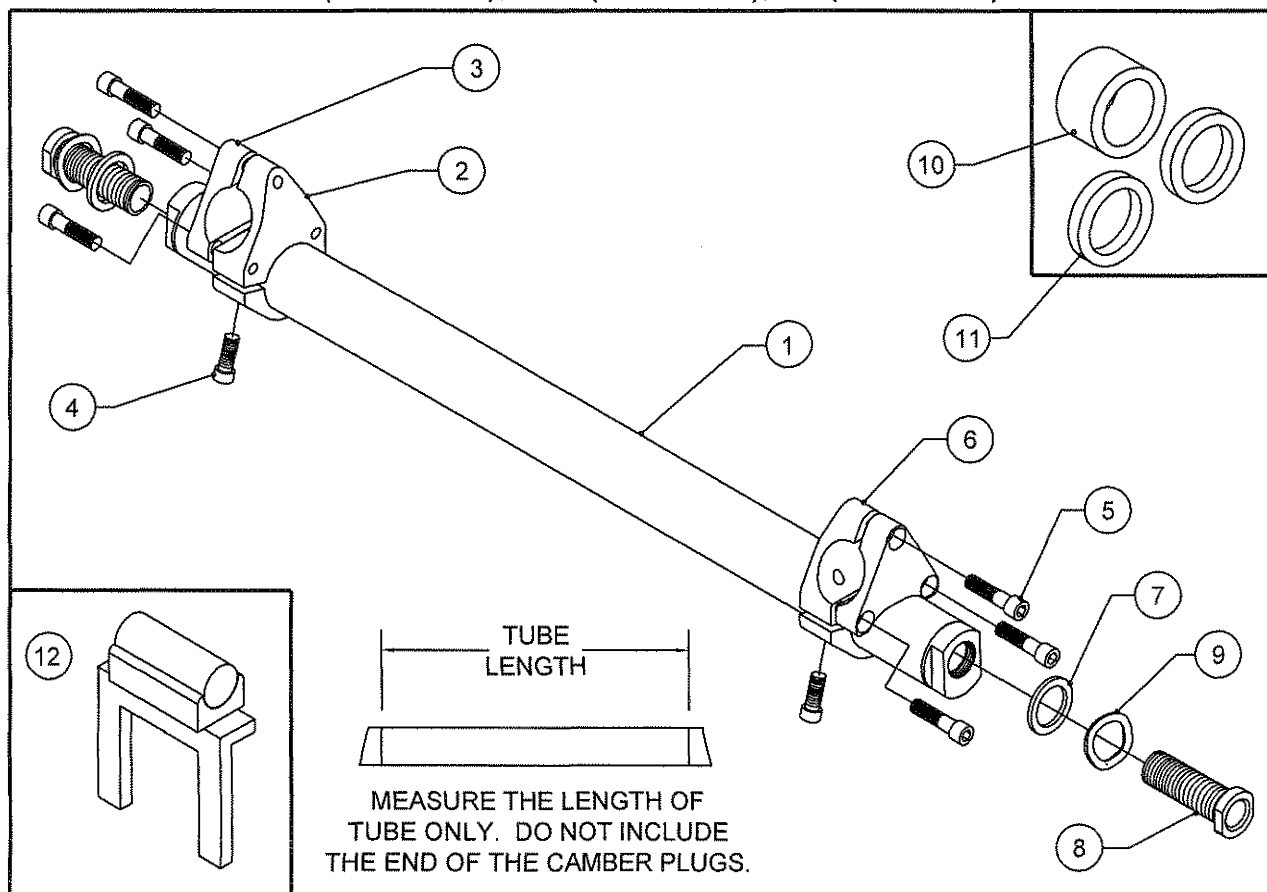

ITEM	QUANTITY	PART NO.	DESCRIPTION
CON. 1	1	TABLE ONE	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18°, CAMBER TUBE
2	1	CP100258	CAMBER TUBE MOUNT 1.25" TO 1" LEFT
3	2	CP100259	AXLE CLAMP 1"
4	2	HD100050	1/4-20 X 3/4" SOCKET HEAD CAP SCREW
5	6	HD100051	1/4-20 X 1" SOCKET HEAD CAP SCREW
6	1	CP100257	CAMBER TUBE MOUNT 1.25" TO 1" RIGHT
7	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .036 THICK
8	2	CP100151	AXLE SLEEVE, QUICK RELEASE
9	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
10	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
11	4	HD100089	WASHER .906" X .638" X .125
12	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
7-9		AS116180	AXLE SLEEVE PACKAGE
10-11		AS100945	REAR WHEEL SPACING PACKAGE
CON. 1-9		TABLE TWO	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18°, CAMBER TUBE ASSY

TABLE ONE

° CAMBER	PART NO.
0°	CP100419
2°	CP100420
4°	CP100421
6°	CP100999
8°	CP100422
12°	CP100423
15°	CP100424
18°	CP100425


TABLE TWO

° CAMBER	ASSY NO.
0°	AS100053
2°	AS100059
4°	AS100054
6°	AS100592
8°	AS100055
12°	AS100056
15°	AS100076
18°	AS100077

CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING

AMPUTEE AXLE PLATE ASSEMBLY

TiLite™ TR, YR, TR(SERIES 2), YR(SERIES 2), TR (SERIES 3)

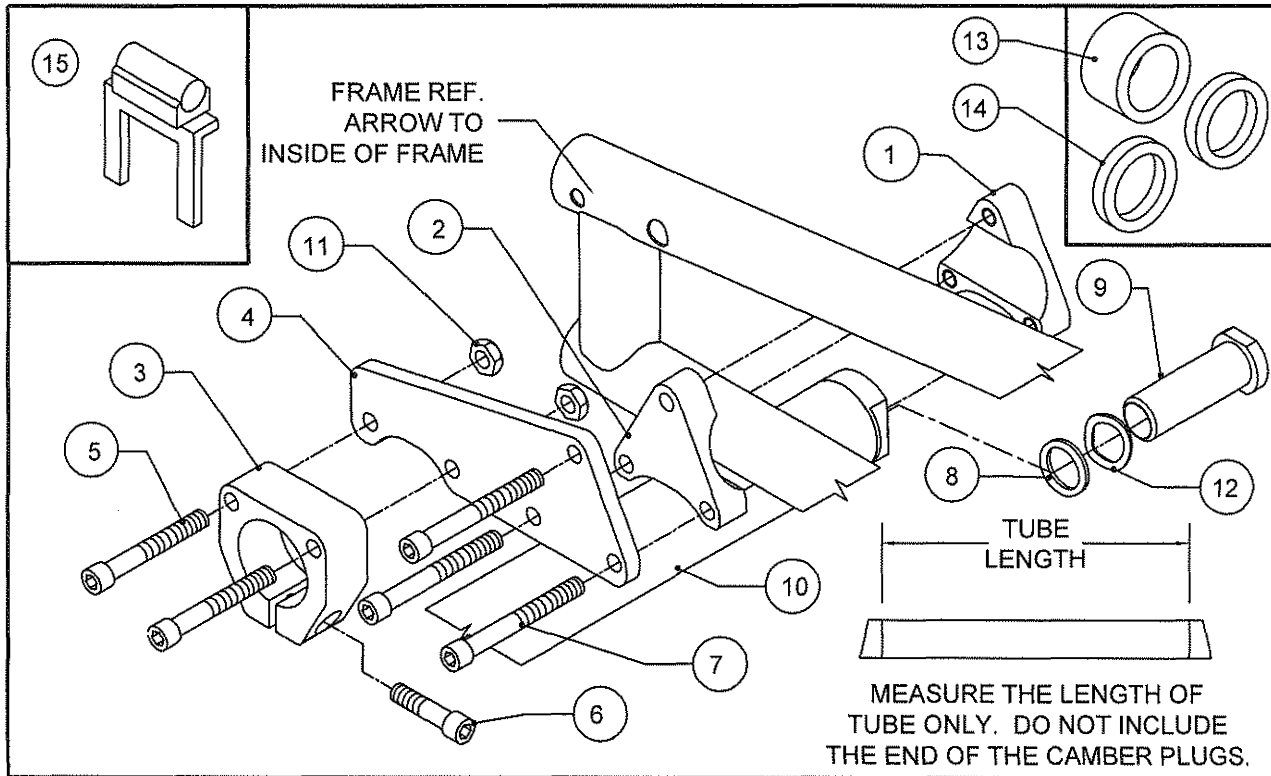

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101061	CAMBER MOUNT THREADED
2	2	CP101060	CAMBER MOUNT NON-THREADED
3	2	CP101063	ADJUSTABLE HEIGHT CAMBER TUBE MOUNT
4	2	CP101197	AMPUTEE AXLE PLATE
5A	4	HD100023	1/4-20 X 1-1/4 LONG SOCKET HEAD CAP SCREW
5B	4	CP101045	1/4-20 X 2-1/4 LONG SOCKET HEAD CAP SCREW
6	2	HD100051	1/4-20 X 1" SOCKET HEAD CAP SCREW
7	6	HD100055	1/4-20 X 1-3/4 SOCKET HEAD CAP SCREW
8	2	HD100072	STEEL WASHER, I.D. .630 X .868 O.D. X .036 THICK
9	2	CP100151	AXLE SLEEVE, QUICK RELEASE, .500 I.D.; 5/8-18 UNF-2A, 2" LONG
10	2	TABLE ONE	0°, 2°, 4°, 6°, 8°, 12°, 15°, 18° CAMBER TUBE
11	4	HD100049	1/4-20 NYLOCK LOCKNUT THIN STYLE
12	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
13	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
14	4	HD100089	WASHER .906" X .638" X .125
15	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
8-9, 12		AS116180	AXLE SLEEVE PACKAGE
13-14		AS100945	REAR WHEEL SPACING PACKAGE
1-5A,6-9,11,12		AS101056	AMPUTEE AXLE PLATE ASSY, PAIR (W/O CAMBERTUBE)
1-5B,6-9,11,12		AS102796	AMPUTEE AXLE PLATE ASSY FOR ANTITIP, PAIR (W/O CAMBERTUBE)
1-5A,6-12		TABLE TWO	AMPUTEE AXLE PLATE ASSY, PAIR
1-5B,6-12		TABLE THREE	AMPUTEE AXLE PLATE W/ANTITIP ASSY, PAIR

TABLE ONE

° CAMBER	PART NO.
0°	CP100419
2°	CP100420
4°	CP100421
6°	CP100999
8°	CP100422
12°	CP100423
15°	CP100424
18°	CP100425

TABLE TWO


° CAMBER	PART NO.
0°	AS101048
2°	AS101049
4°	AS101050
6°	AS101051
8°	AS101052
12°	AS101053
15°	AS101054
18°	AS101055

TABLE THREE

° CAMBER	PART NO.
0°	AS102797
2°	AS102798
4°	AS102799
6°	AS102800
8°	AS102801
12°	AS102802
15°	AS102803
18°	AS102804

CON.= CONSULT FACTORY FOR TUBE LENGTH WHEN ORDERING


TiLite™ TR(SERIES 2), TRC(SERIES 2), YR(SERIES 2)

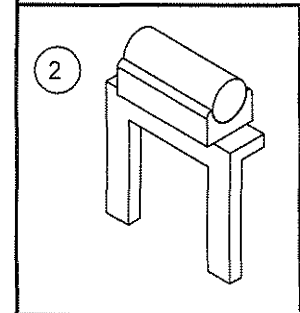

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP103126	FROG LEG SUSPENSION - USERS UNDER 180 LBS.
1B	1	CP103127	FROG LEG SUSPENSION - USERS OVER 180 LBS.
2	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER

AXLE ASSEMBLY


TiLite™ TR (SERIES 3)

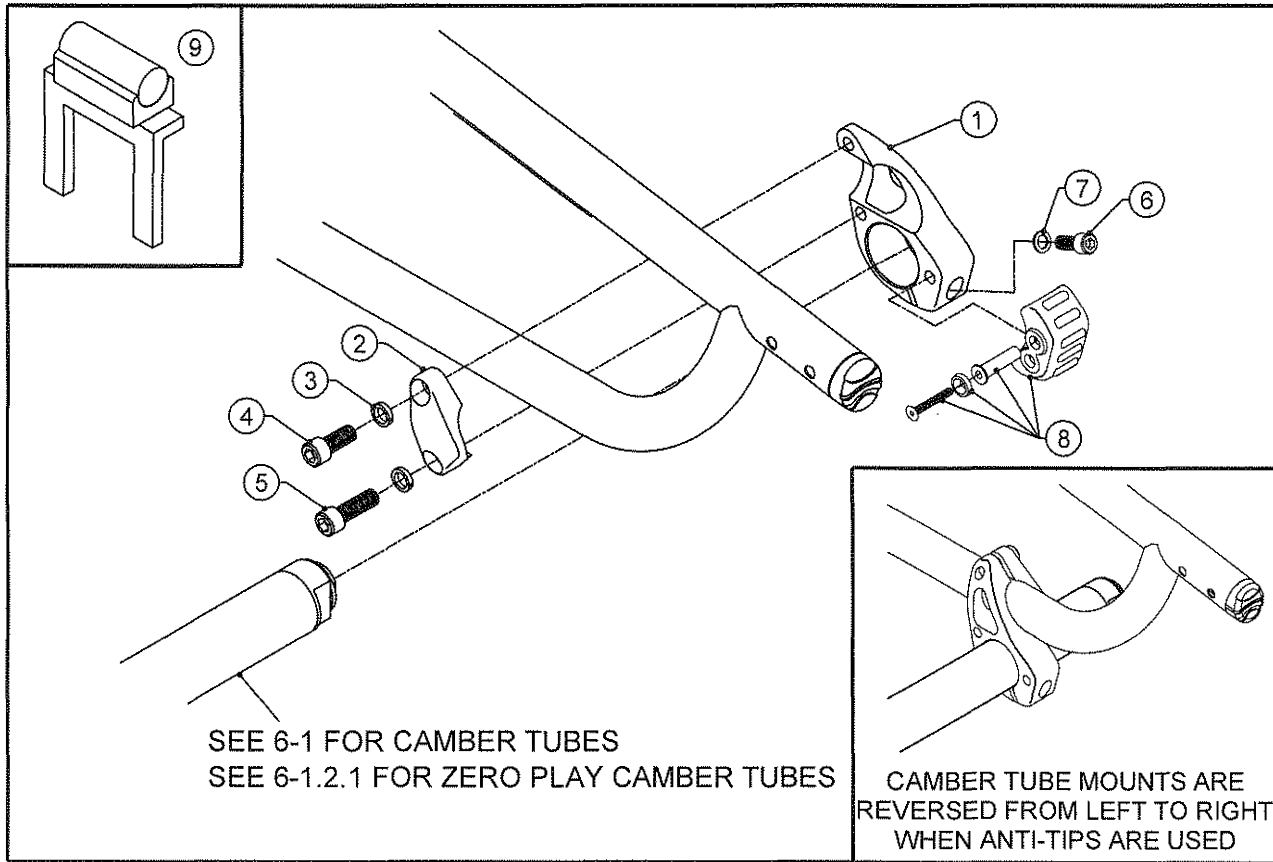

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP106259	CAMBER TUBE MOUNT RIGHT
1B	1	CP106260	CAMBER TUBE MOUNT LEFT
2A	1	CP106257	CAMBER TUBE MOUNT TOP RIGHT
2B	1	CP106258	CAMBER TUBE MOUNT TOP LEFT
3	4	HD100695	5/16 HI-COLLAR LOCK WASHER
4	2	HD100693	5/16-18 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
5	2	HD100694	5/16-18 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC
6	2	HD100458	1/4-20 X 5/8" SOCKET HEAD CAP SCREW, BLACK ZINC
7	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL
8	2	AS110941	CAMBER MOUNT BUMPER ASSY
9	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1A,1B,2A,2B,3-8		AS115101	SERIES 3 AXLE ASSEMBLY, PAIR
1A,1B,2A,2B,3-7		AS115102	SERIES 3 AXLE ASSEMBLY FOR ANTI TIPS, PAIR

AXLE ASSEMBLY

TiLite™ ZRA, AERO Z

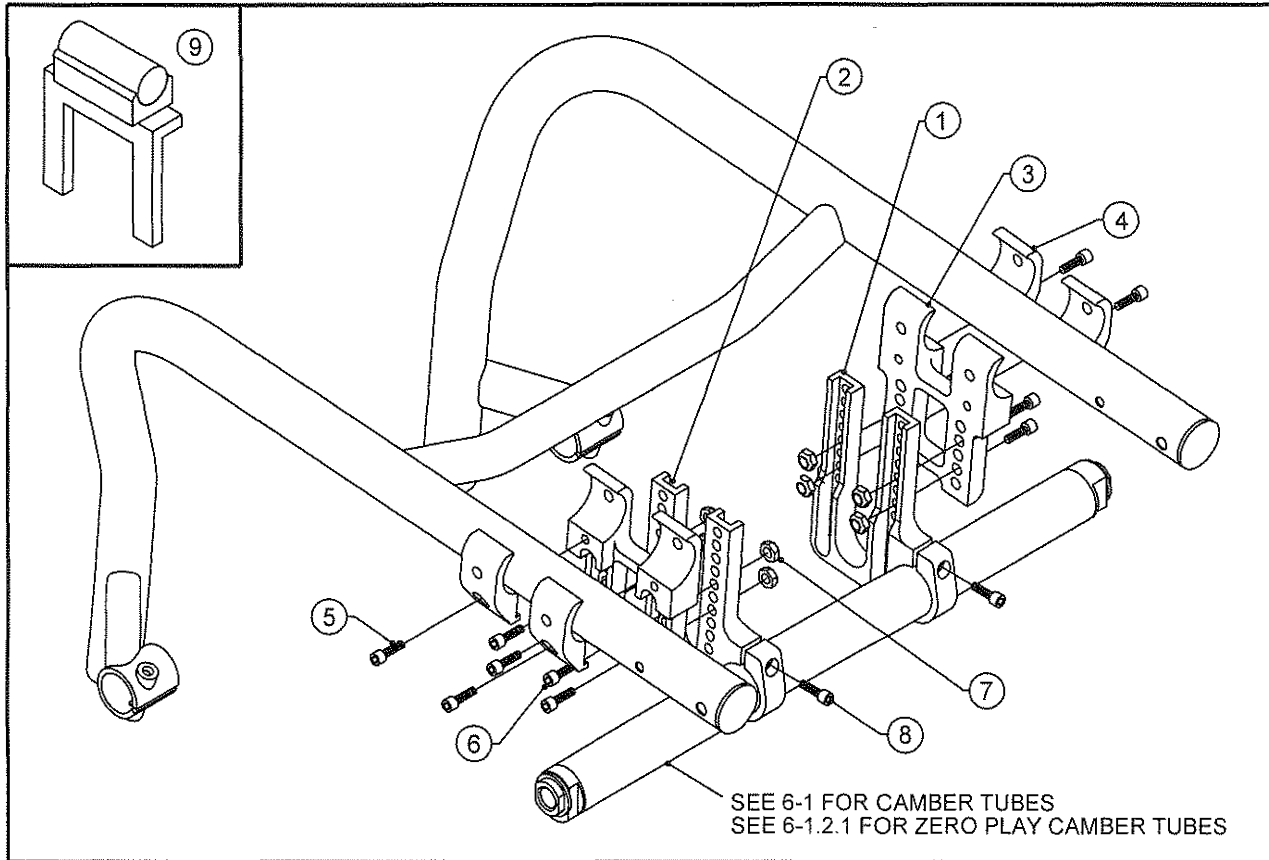


ITEM	QUANTITY	PART NO.	DESCRIPTION
*1	1	CP102050	CAMBERTUBE MOUNT MONOTUBE RIGID RT
*2	1	CP102060	CAMBERTUBE MOUNT MONOTUBE RIGID LT
3	1	CP102293	CAMBER MOUNT WEBBED
4	2	CP102296	TOE CLAMP 1 1/4 CAMBER MOUNT WITH HOLE
5	4	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
6	8	HD100327	1/4-28 X 3/4" SOCKET HEAD CAP SCREW, BLACK
7	8	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
8	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
9	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1-8		AS102820	AXLE ASSEMBLY W/ WEBBED CAMBER MOUNT

*NOTE: NEED SERIAL NUMBER OF CHAIR WHEN ORDERING, PARTS ARE FIT TO CHAIR.

AXLE ASSEMBLY


TiLite™ ZRA (SERIES 2), AERO Z (SERIES 2)

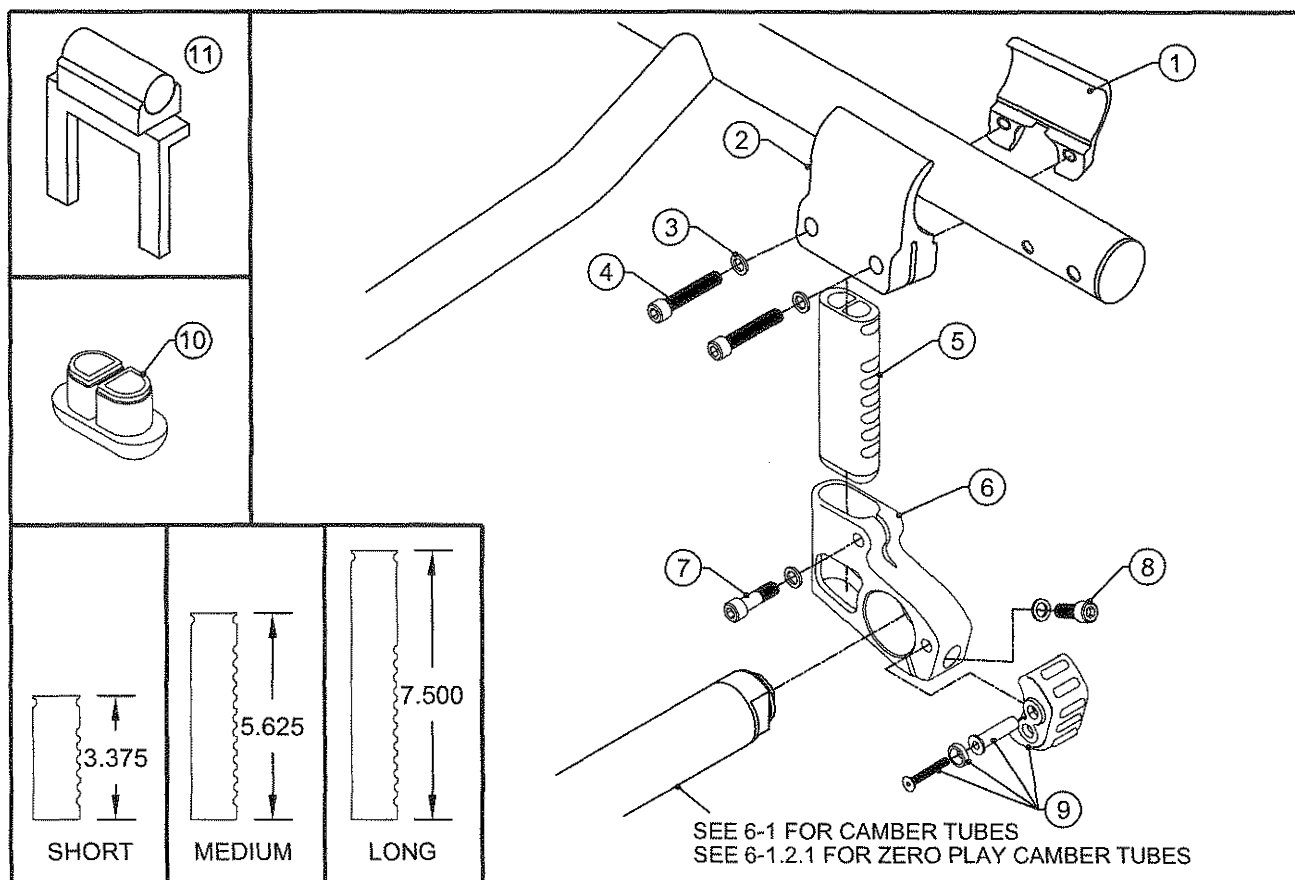

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP107408	Z2 OUTSIDE CAMBER MOUNT CLAMP FOR RAIL
2	2	CP107410	Z2 INSIDE CAMBER MOUNT CLAMP FOR RAIL
3	8	HD100348	WASHER .255ID X .395OD X .048THK STEEL
4	4	HD100607	1/4-20 X 1 3/8" SOCKET HEAD CAP SCREW BLACK ZINC
5A, (10)	2	AS111197	VERTICAL STRUT SHORT SUB-ASSY (WITH ENDCAP)
5B, (10)	2	AS111198	VERTICAL STRUT MEDIUM SUB-ASSY (WITH ENDCAP)
5C, (10)	2	AS111199	VERTICAL STRUT LONG SUB-ASSY (WITH ENDCAP)
6A	1	CP104336	CAMBER TUBE CLAMP, RIGHT
6B	1	CP104337	CAMBER TUBE CLAMP, LEFT
7	2	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW BLACK ZINC
8	2	HD100458	1/4-20 X 5/8" SOCKET HEAD CAP SCREW, BLACK ZINC
9	2	AS110941	CAMBER MOUNT BUMPER ASSY
10	2	CP104350	CAMBER MOUNT EXTENSION ENDCAP (REPLACEMENT FOR VERTICAL STRUT)
11	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1-4,5A,6A,6B-9,(10)		AS117124	AXLE ASSEMBLY W/ SHORT STRUT, PAIR
1-4,5B,6A,6B-9,(10)		AS117125	AXLE ASSEMBLY W/ MEDIUM STRUT, PAIR
1-4,5C,6A,6B-9,(10)		AS117126	AXLE ASSEMBLY W/ LONG STRUT, PAIR
1-4,5A,6A,6B-8,(10)		AS117127	AXLE ASSY W/ SHORT STRUT FOR ANTI-TIPS, PAIR (BUMPER NOT INCLUDED)
1-4,5B,6A,6B-8,(10)		AS117128	AXLE ASSY W/ MEDIUM STRUT FOR ANTI-TIPS, PAIR (BUMPER NOT INCLUDED)
1-4,5C,6A,6B-8,(10)		AS117129	AXLE ASSY W/ LONG STRUT FOR ANTI-TIPS, PAIR (BUMPER NOT INCLUDED)

AXLE ASSEMBLY

TiLite™ TWIST, AERO T, TRA (SERIES 2)

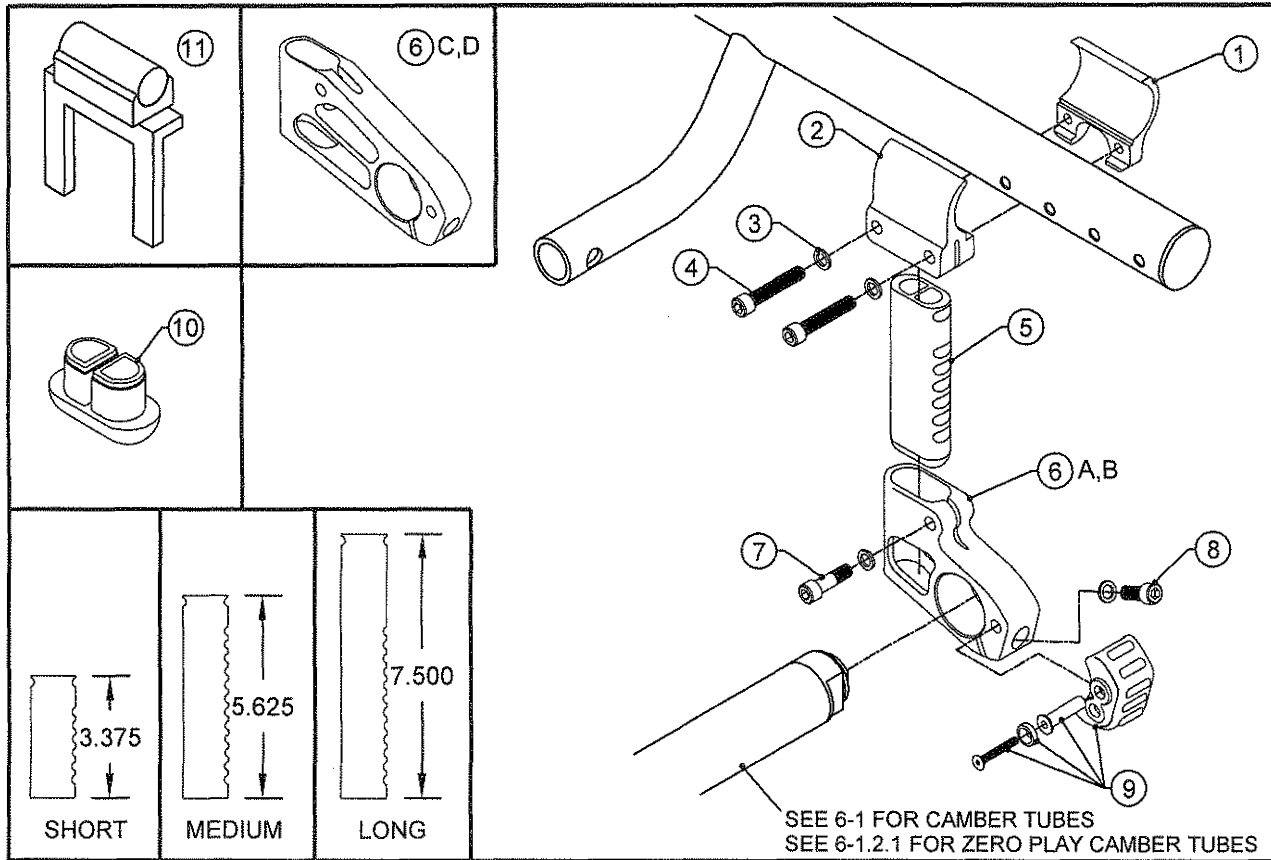


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP105111	PED OUTSIDE CLAMP
2	2	CP105112	PED INSIDE CLAMP
3	8	HD100348	WASHER .255ID X .395OD X .048THK STEEL
4	4	HD100607	1/4-20 X 1 3/8" SOCKET HEAD CAP SCREW BLACK ZINC
5A, (10)	2	AS111197	VERTICAL STRUT SHORT SUB-ASSY (WITH ENDCAP)
5B, (10)	2	AS111198	VERTICAL STRUT MEDIUM SUB-ASSY (WITH ENDCAP)
5C, (10)	2	AS111199	VERTICAL STRUT LONG SUB-ASSY (WITH ENDCAP)
6A	1	CP104336	CAMBER TUBE CLAMP, RIGHT
6B	1	CP104337	CAMBER TUBE CLAMP, LEFT
6C	1	CP107119	CAMBER TUBE CLAMP EXTENDED RIGHT
6D	1	CP107120	CAMBER TUBE CLAMP EXTENDED LEFT
7	2	HD100608	1/4-20 X 7/8" SOCKET HEAD CAP SCREW BLACK ZINC
8	2	HD100458	1/4-20 X 5/8" SOCKET HEAD CAP SCREW, BLACK ZINC
9	2	AS110941	CAMBER MOUNT BUMPER ASSY
10	2	CP104350	CAMBER MOUNT EXTENSION ENDCAP (REPLACEMENT FOR VERTICAL STRUT)
11	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1-4, 5A, 6A, 6B-9, (10)		AS111761	AXLE ASSEMBLY W/ SHORT STRUT, PAIR
1-4, 5B, 6A, 6B-9, (10)		AS111762	AXLE ASSEMBLY W/ MEDIUM STRUT, PAIR
1-4, 5C, 6A, 6B-9, (10)		AS111763	AXLE ASSEMBLY W/ LONG STRUT, PAIR
1-4, 5A, 6A, 6B-8, (10)		AS111764	AXLE ASSY W/ SHORT STRUT FOR ANTI-TIPS, PAIR (BUMPER NOT INCLUDED)
1-4, 5B, 6A, 6B-8, (10)		AS111765	AXLE ASSY W/ MEDIUM STRUT FOR ANTI-TIPS, PAIR (BUMPER NOT INCLUDED)
1-4, 5C, 6A, 6B-8, (10)		AS111766	AXLE ASSY W/ LONG STRUT FOR ANTI-TIPS, PAIR (BUMPER NOT INCLUDED)
1-4, 5A, 6C, 6D-9, (10)		AS115911	EXTENDED AXLE ASSEMBLY W/ SHORT STRUT, PAIR
1-4, 5B, 6C, 6D-9, (10)		AS115912	EXTENDED AXLE ASSEMBLY W/ MEDIUM STRUT, PAIR
1-4, 5C, 6C, 6D-9, (10)		AS115913	EXTENDED AXLE ASSEMBLY W/ LONG STRUT, PAIR
1-4, 5A, 6C, 6D-8, (10)		AS115914	EXT AXLE ASSY W/ SHORT STRUT FOR ANTI-TIPS, PAIR (BUMPER N/INCLUDED)
1-4, 5B, 6C, 6D-8, (10)		AS115915	EXT AXLE ASSY W/ MEDIUM STRUT FOR ANTI-TIPS, PAIR (BUMPER N/INCLUDED)
1-4, 5C, 6C, 6D-8, (10)		AS115916	EXT AXLE ASSY W/ LONG STRUT FOR ANTI-TIPS, PAIR (BUMPER N/INCLUDED)



AXLE ASSEMBLY


TiLite™ ZR, Z10

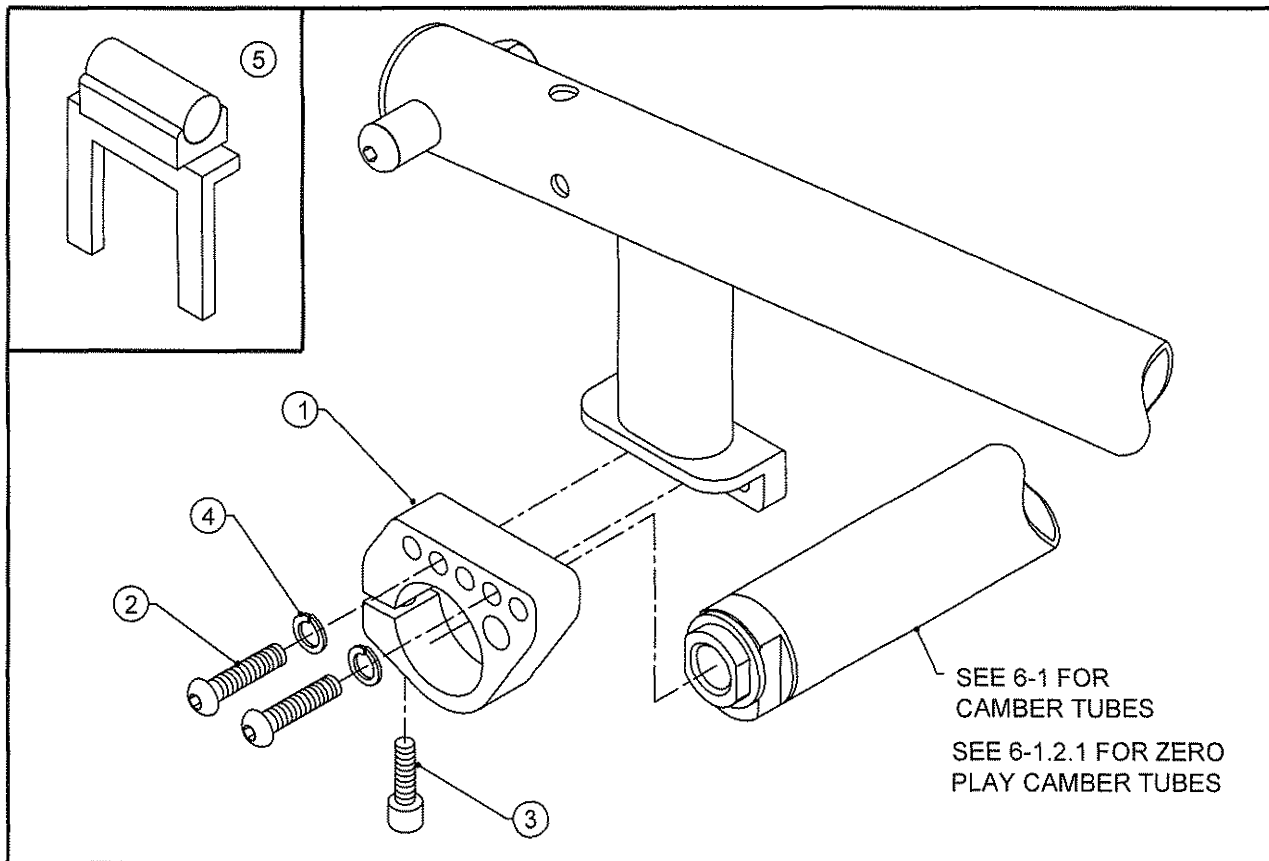

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP102280	CAMBER CLAMP
2	4	HD100080	1/4-28 X 1" BUTTON HEAD CAP SCREW
3	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
4	4	HD100156	1/4" LOCK WASHER
5	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1-4		AS102810	AXLE PLATE ASSEMBLY, PAIR

AXLE ASSEMBLY


TiLite™ ZR (SERIES 2)

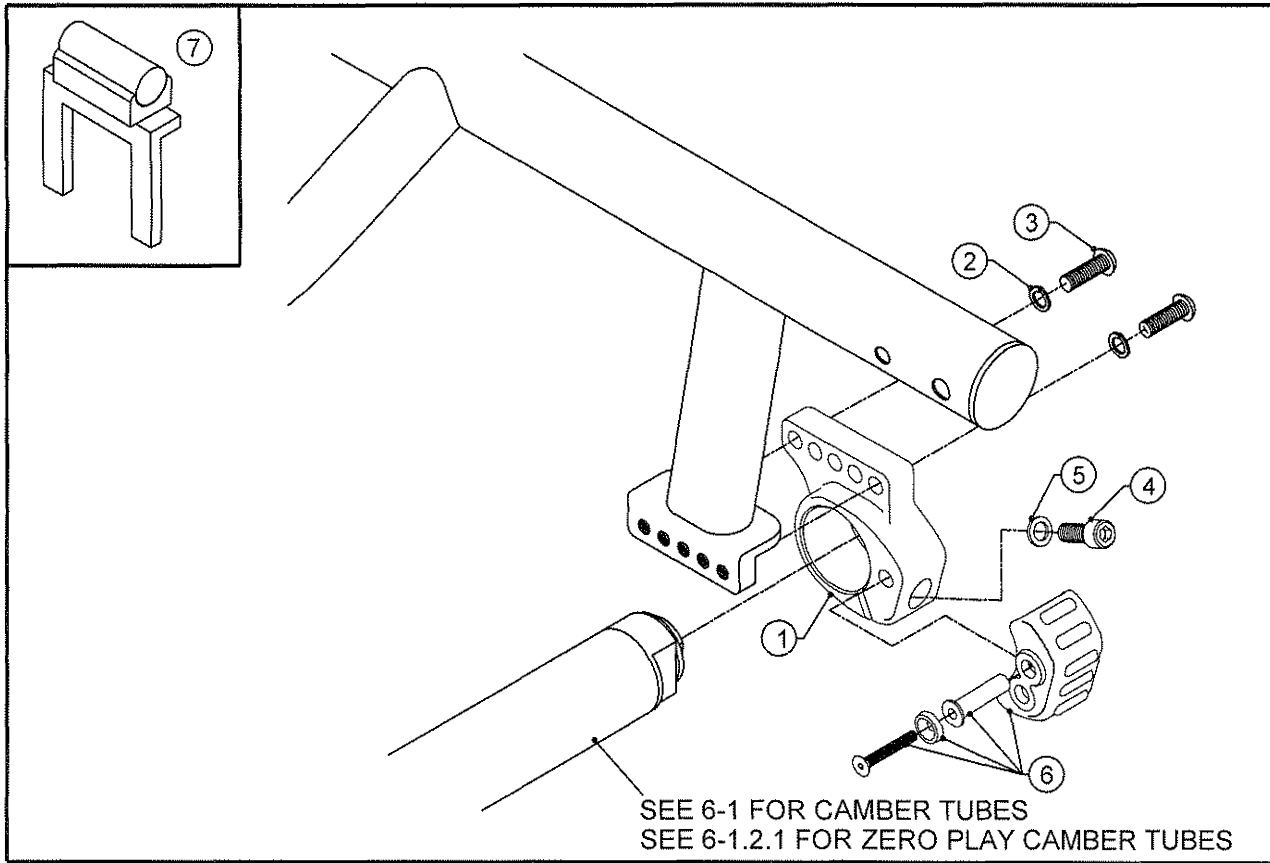

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104373	CAMBER TUBE CLAMP, RIGHT
1B	1	CP104374	CAMBER TUBE CLAMP, LEFT
2	4	HD100618	1/4" LOCK WASHER, BLACK ZINC
3	4	HD100408	1/4-28 X 7/8" BUTTON HEAD CAP SCREW BLACK ZINC
4	2	HD100458	1/4-20 X 5/8" SOCKET HEAD CAP SCREW, BLACK ZINC
5	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, BLACK ZINC PLATED
6		AS110941	CAMBER MOUNT BUMPER ASSY
7	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1A,1B,2-6		AS110826	AXLE ASSEMBLY, PAIR
1A,1B,2-5		AS110827	AXLE ASSEMBLY FOR ANTI TIPS, PAIR (BUMPER NOT INCLUDED)

AXLE ASSEMBLY


TiLite™ Aero R

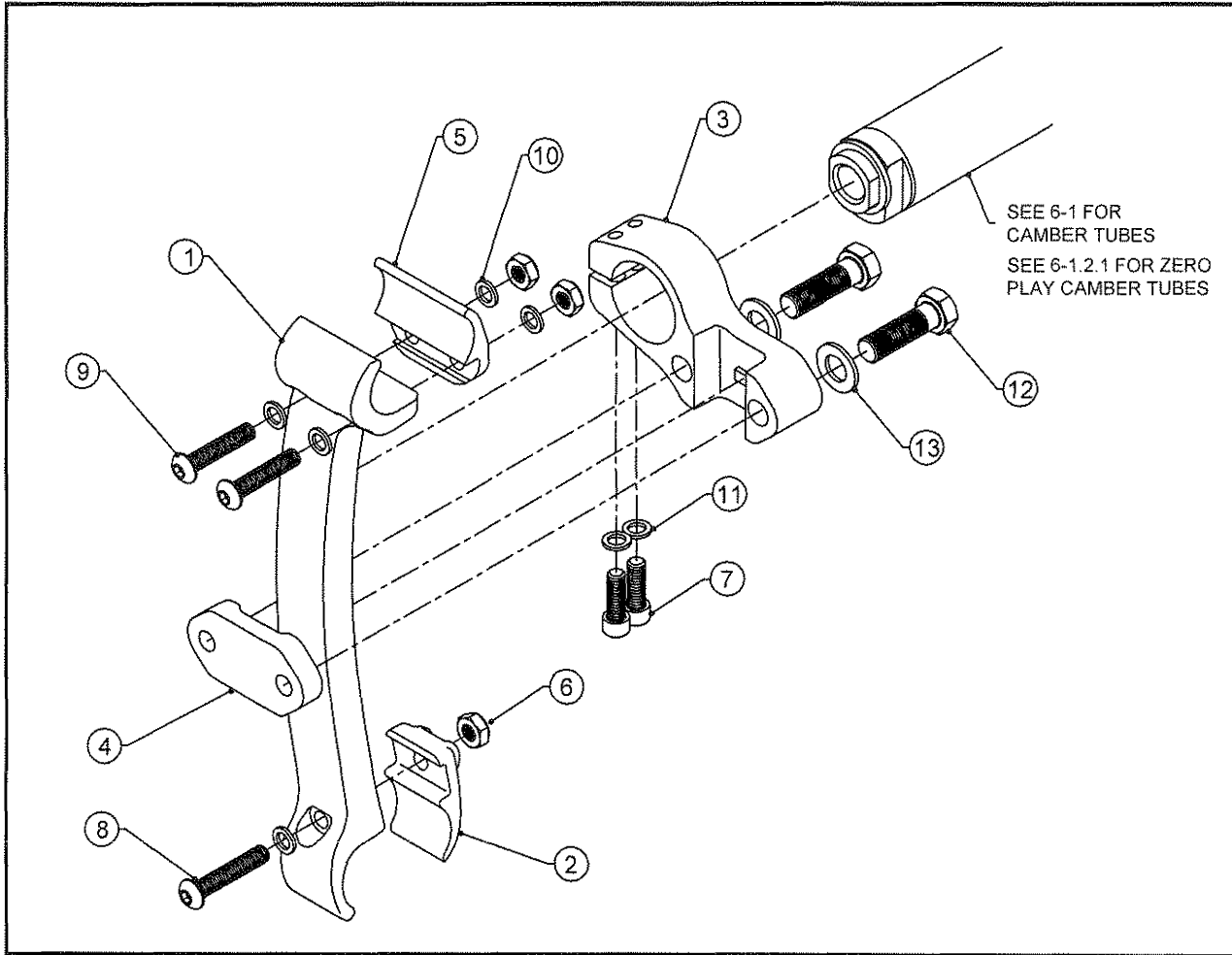

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP102754	DIE CAST AXLE PLATE RIGHT
1B	1	CP102755	DIE CAST AXLE PLATE LEFT
2A	1	CP102756	DIE CAST TOE CLAMP RIGHT
2B	1	CP102757	DIE CAST TOE CLAMP LEFT
3A	1	CP103208	CAMBER CLAMP RIGHT HAND
3B	1	CP103209	CAMBER CLAMP LEFT HAND
4	2	CP103200	CAMBER PLATE CLAMP
5	2	CP103130	RIGID AXLE PLATE CLAMP
6	6	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
7	2	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
8	2	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
9	4	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
10	4	HD100056	1/4" AN WASHER .260 ID .500 UOD .0625 THICK, YELLOW ZINC PLATED
11	10	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	2	HD100470	3/8-16 X 1-1/2" GRADE 8 HEX HEAD CAP SCREW, BLACK ZINC
13	2	HD100353	3/8" AN WASHER STAINLESS STEEL
1-13		AS104157	AXLE PLATE ASSEMBLY, PAIR

PEDIATRIC CHAIR AXLE ASSEMBLY


TiLite™ YG, YGS

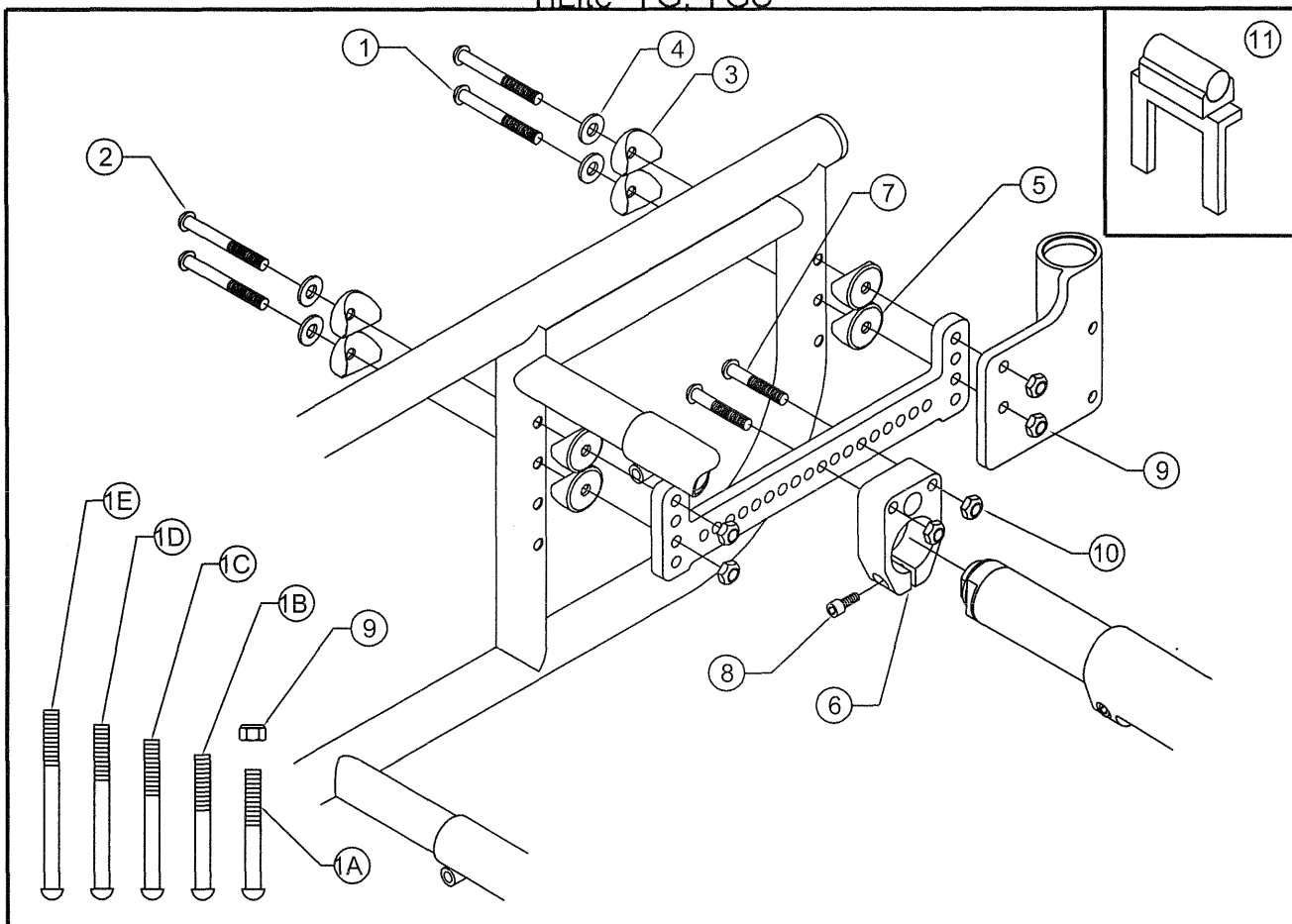

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	4	HD100104	1/4-28 X 2" BUTTON HEAD CAP SCREW
1B	4	HD100105	1/4-28 X 2-1/4" BUTTON HEAD CAP SCREW
1C	4	HD100245	1/4-28 X 2-1/2" BUTTON HEAD CAP SCREW
1D	4	HD100351	1/4-28 X 2-3/4" BUTTON HEAD CAP SCREW
1E	4	HD100598	1/4-28 X 3" BUTTON HEAD CAP SCREW
2	4	HD100104	1/4-28 X 2" BUTTON HEAD CAP SCREW
3	16	HD100014	COVER WASHER 1" O.D. X 1/4" I.D. X 1/8" THK
4	8	HD100022	1/4 SAE FLAT WASHER
5	2	CP101871	PEDIATRIC ADJUSTABLE AXLE PLATE
6	2	CP101868	PEDIATRIC CHAIR CAMBER CLAMP ADJUSTABLE
7	4	HD100123	1/4-20 X 1-1/2" BUTTON HEAD CAP SCREW
8	2	HD100050	1/4-20 X 3/4" SOCKET HEAD CAP SCREW
9	8	HD100021	1/4-28 X .218 NYLOCK LOCKNUT, CLEAR ZINC PLATED (thin)
10	4	HD100049	1/4-20 NYLOCK LOCKNUT THIN STYLE
11	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1A,2-10		AS102031	PEDIATRIC AXLE ASSEMBLY, PAIR
1B,2-10		AS102032	PEDIATRIC AXLE ASSEMBLY FOR STROLLER HANDLES, PAIR
1C,2-10		AS102780	PEDIATRIC AXLE ASSY FOR STROLLER HANDLES W/ ANTI-TIP, PAIR
1A-E,9		AS102774	ACCESSORY BOLT PACKAGE

AXLE PLATE ASSEMBLY


TiLite™ TRA, YRA

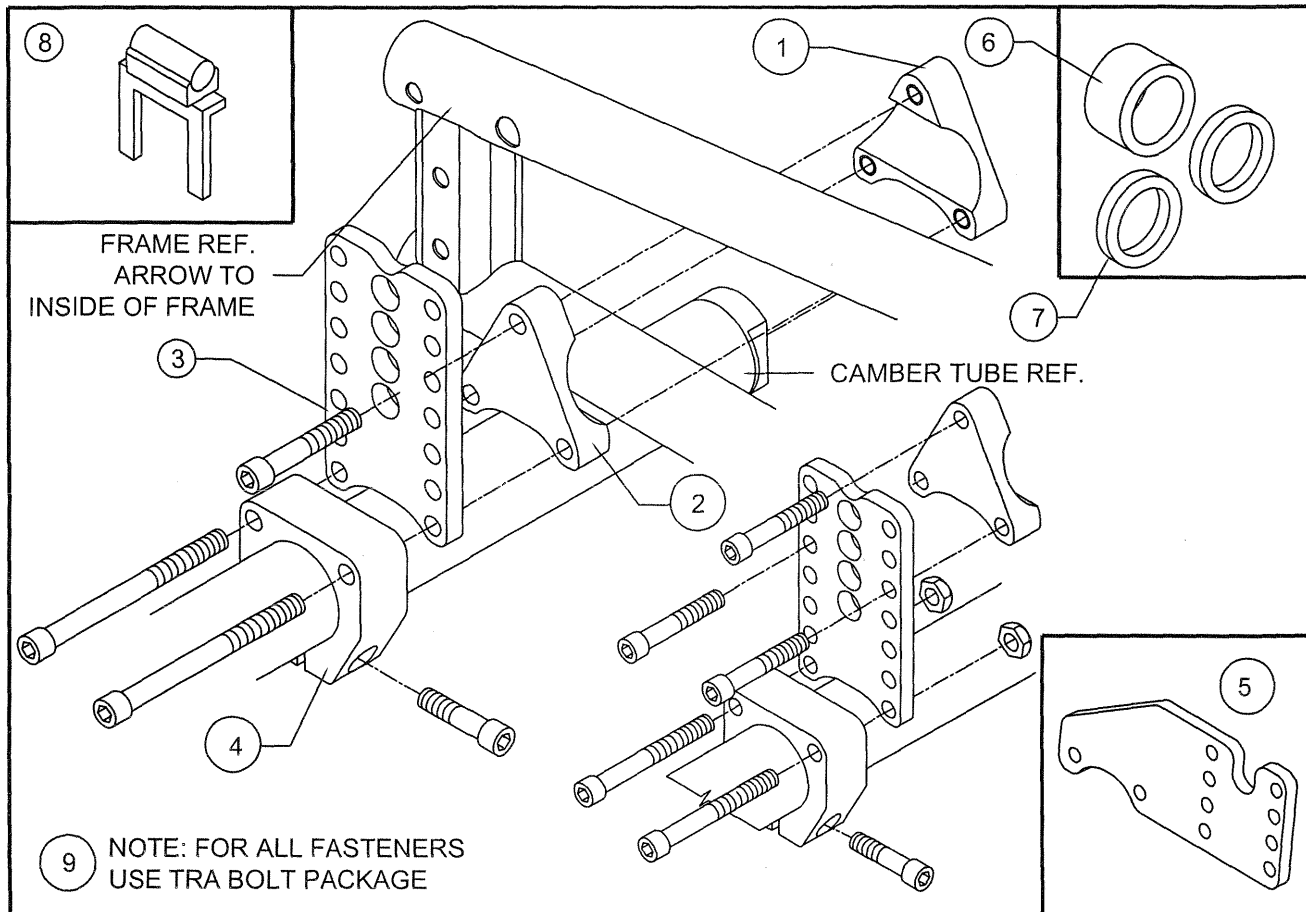

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101061	CAMBER MOUNT THREADED
2	2	CP101060	CAMBER MOUNT NON-THREADED
3	2	CP101845	ADJ HEIGHT CAMBER TUBE PLATE
4	2	CP101063	ADJ HEIGHT CAMBER TUBE MOUNT
5	2	CP101169	ADJUSTABLE HEIGHT CAMBER TUBE PLATE, AMPUTEE
6	2	CP100962	SPINERIGY WHEEL AXLE SLEEVE SPACER
7	4	HD100089	WASHER .906" X .638" X .125
8	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
6-7		AS100945	REAR WHEEL SPACING PACKAGE
1-4		AS101917	AXLE PLATE ASSEMBLY, PAIR
1, 2, 4-5		AS100959	AMPUTEE AXLE PLATE ASSY, PAIR
9		AS100958	BOLT PACKAGE FOR AXLE (NOTE CONTAINS ALL HARDWARE NEEDED FOR REAR AXLE ASSEMBLIES)

TRA AXLE PLATE ASSEMBLY (DISC 1/27/03)


TiLite™ TRA

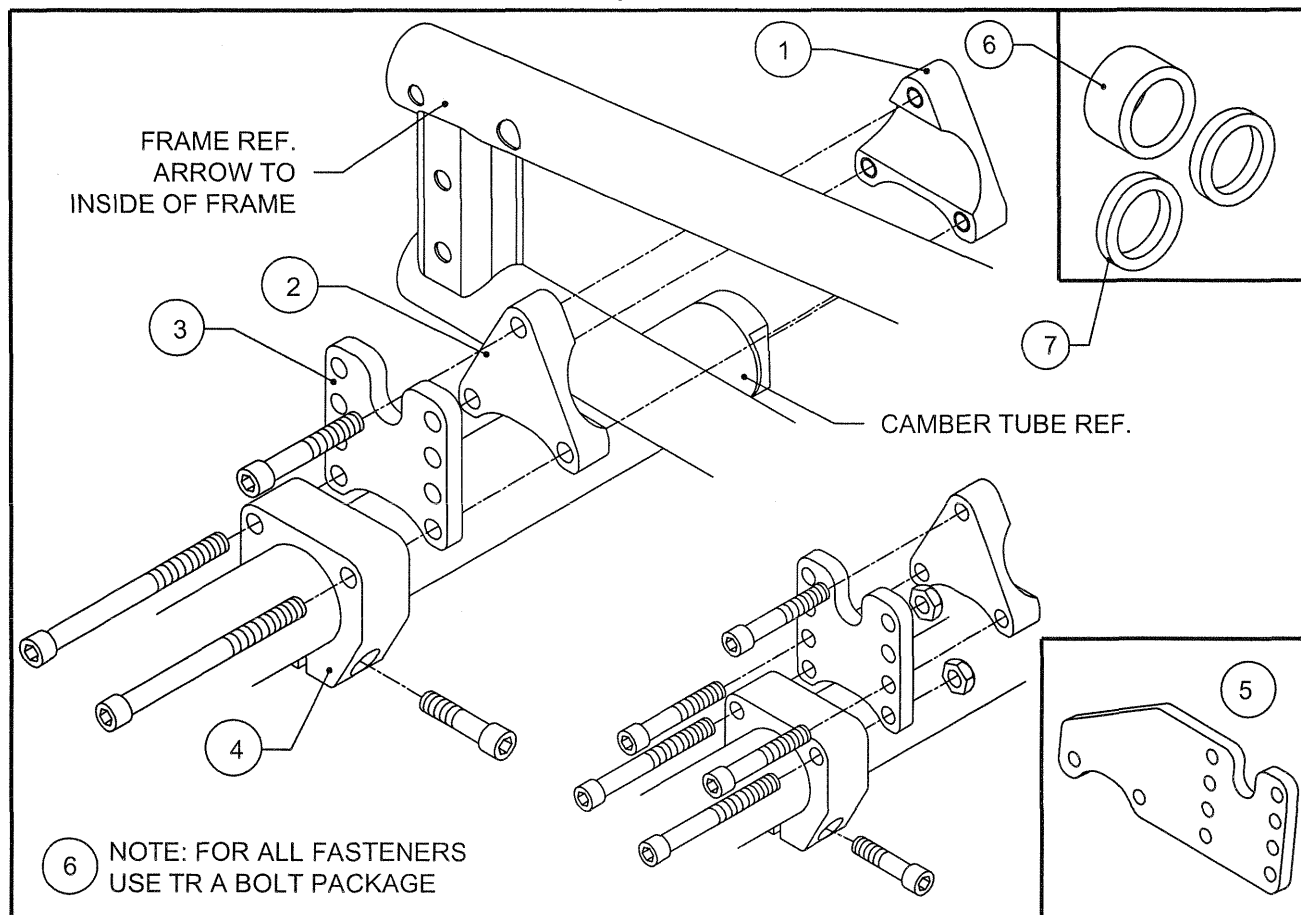

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101061	CAMBER MOUNT THREADED
2	2	CP101060	CAMBER MOUNT NON-THREADED
3	2	CP101062	ADJ HEIGHT CAMBER TUBE PLATE
4	2	CP101063	ADJ HEIGHT CAMBER TUBE MOUNT
5	2	CP101169	ADJUSTABLE HEIGHT CAMBER TUBE PLATE, AMPUTEE
6	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
7	4	HD100089	WASHER .906" X .638" X .125
6-7		AS100945	TR REAR WHEEL SPACING PACKAGE
1-4		AS100918	AXLE PLATE ASSEMBLY TRA, PAIR
1, 2, 4-5		AS100959	AMPUTEE AXLE PLATE ASSY TRA, PAIR
6		AS100958	TR A BOLT PACKAGE FOR AXLE (NOTE CONTAINS ALL HARDWARE NEEDED FOR REAR AXLE ASSEMBLIES)

AXLE ASSEMBLY


TiLite™ EVO

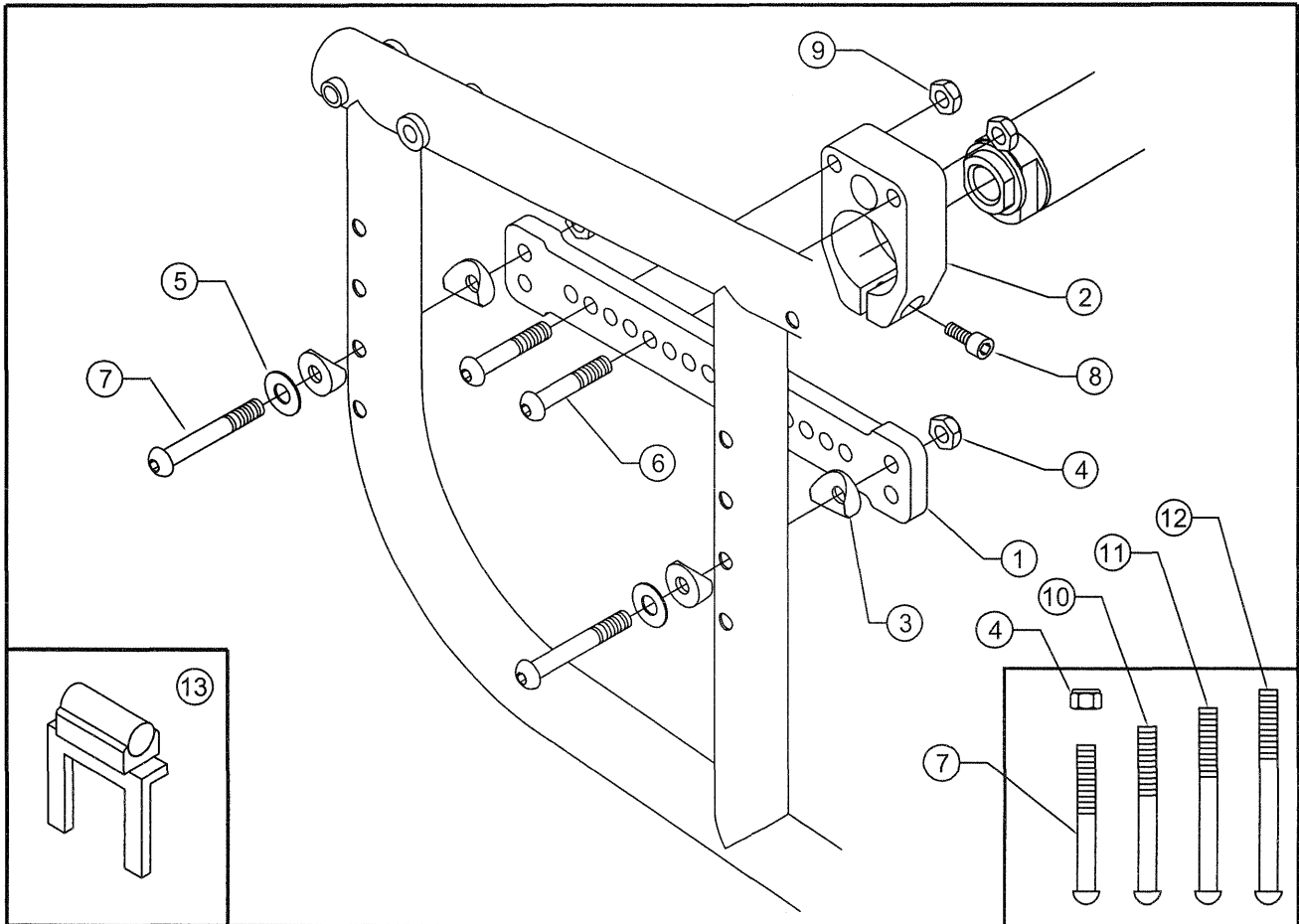

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP101855	RIGID ADJUSTABLE AXLE PLATE
2	2	CP101868	ADJUSTABLE CAMBER CLAMP
3	8	HD100240	CONCAVE WASHER MOLDED
4	4	HD100021	1/4-28 X .218 NYLOCK LOCKNUT
5	4	HD100022	1/4 SAE FLAT WASHER
6	4	HD100123	1/4-20 X 1-1/2" BUTTON HEAD CAP SCREW
7	4	HD100104	1/4-28 X 2" BUTTON HEAD CAP SCREW
8	2	HD100050	1/4-20 X 3/4" SOCKET HEAD CAP SCREW
9	4	HD100049	1/4-20 NYLOCK LOCKNUT THIN STYLE
10	8	HD100105	1/4-28 X 2-1/4" BUTTON HEAD CAP SCREW
11	8	HD100245	1/4-28 X 2-1/2" BUTTON HEAD CAP SCREW
12	8	HD100351	1/4-28 X 2-3/4" BUTTON HEAD CAP SCREW
13	1	CP101985	BUBBLE LEVEL BRACKET FOR STANDARD CAMBER
1-9		AS102164	AXLE ASSEMBLY, PAIR
4,7,10-12		AS102774	ACESSORY BOLT PACKAGE

AXLE ASSEMBLY

TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

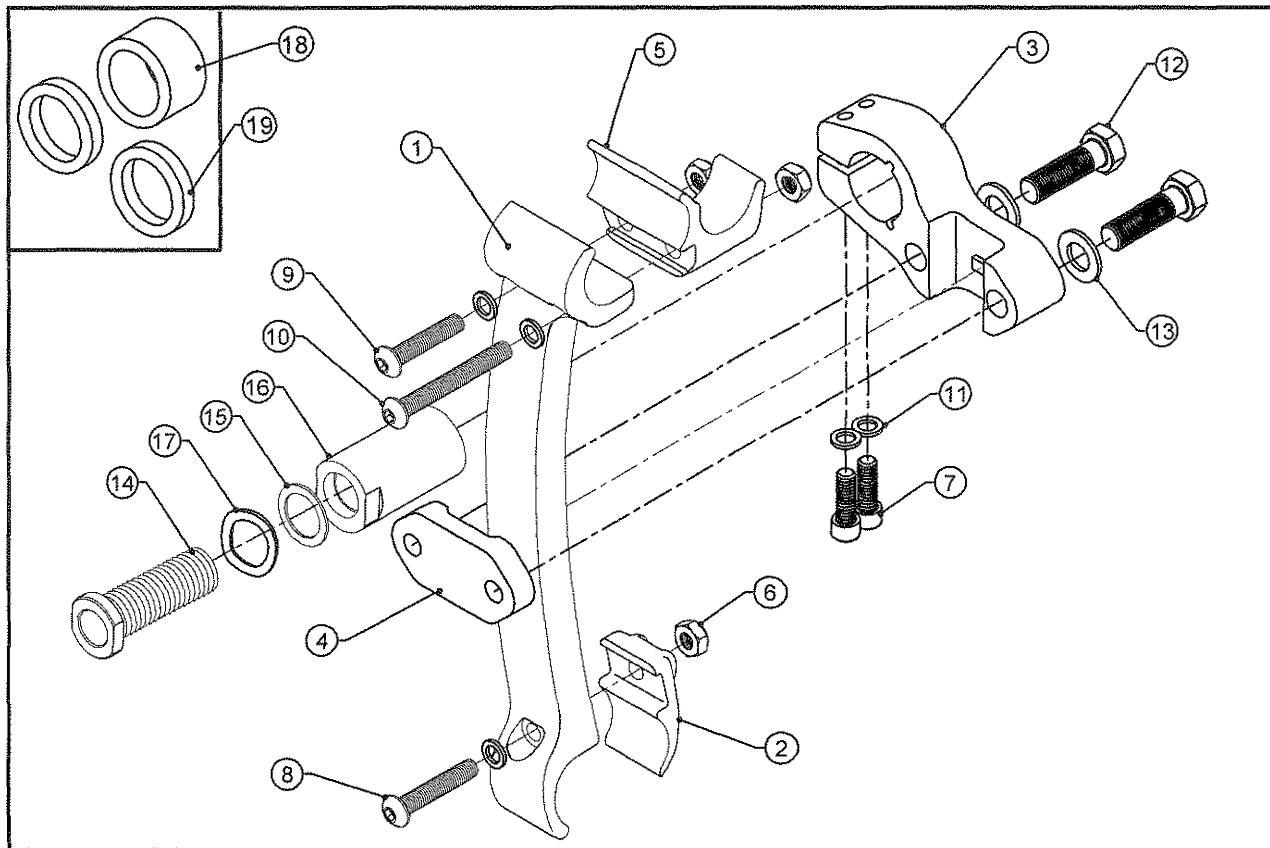


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP102754	DIE CAST AXLE PLATE RIGHT
1B	1	CP102755	DIE CAST AXLE PLATE LEFT
2A	1	CP102756	DIE CAST TOE CLAMP RIGHT
2B	1	CP102757	DIE CAST TOE CLAMP LEFT
3A	1	CP103198	AERO X CAMBER MOUNT RIGHT HAND
3B	1	CP103199	AERO X CAMBER MOUNT LEFT HAND
4	2	CP103200	AERO X / R CAMBER CLAMP
5	2	CP102575	VERTICAL AXLE PLATE COMPOSITE SADDLE CLAMP
6	6	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
7	4	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
8	2	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
9	2	HD100437	1/4-28 X 1-3/8" BUTTON HEAD CAP SCREW, BLACK
10	2	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
11	10	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	4	HD100470	3/8-16 X 1-1/2" GRADE 8 HEX HEAD CAP SCREW, BLACK ZINC
13	4	HD100353	3/8" AN WASHER STAINLESS STEEL
14	1	CP100151	AXLE SLEEVE
15	1	HD100072	STEEL WASHER, .631 I.D X .868 O.D. X .36 THICK
16	1	TABLE ONE	CAMBER PLUG
17	1	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
18	1	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
19	1	HD100089	WASHER .906" X .638" X .125
1-13		AS103236	DIE CAST AXLE PLATE ASSEMBLY, PAIR
14-15, 17		AS116180	AXLE SLEEVE PACKAGE
14-17		TABLE ONE	FOLDER CAMBER ASSEMBLY
18-19		AS100945	REAR WHEEL SPACING PACKAGE

TABLE ONE - CAMBER PLUG

SEAT WIDTH	PART NUMBER	ASSEMBLY NUMBER
0°	CP100365	AS100301
2°	CP100364	AS100302
4°	CP100366	AS100303
6°	CP100867	AS100580
8°	CP100367	AS100304

AMPUTEE AXLE ASSEMBLY

TiLite™ Aero X, 2GX, Aero X (SERIES 2), 2GX (SERIES 2)

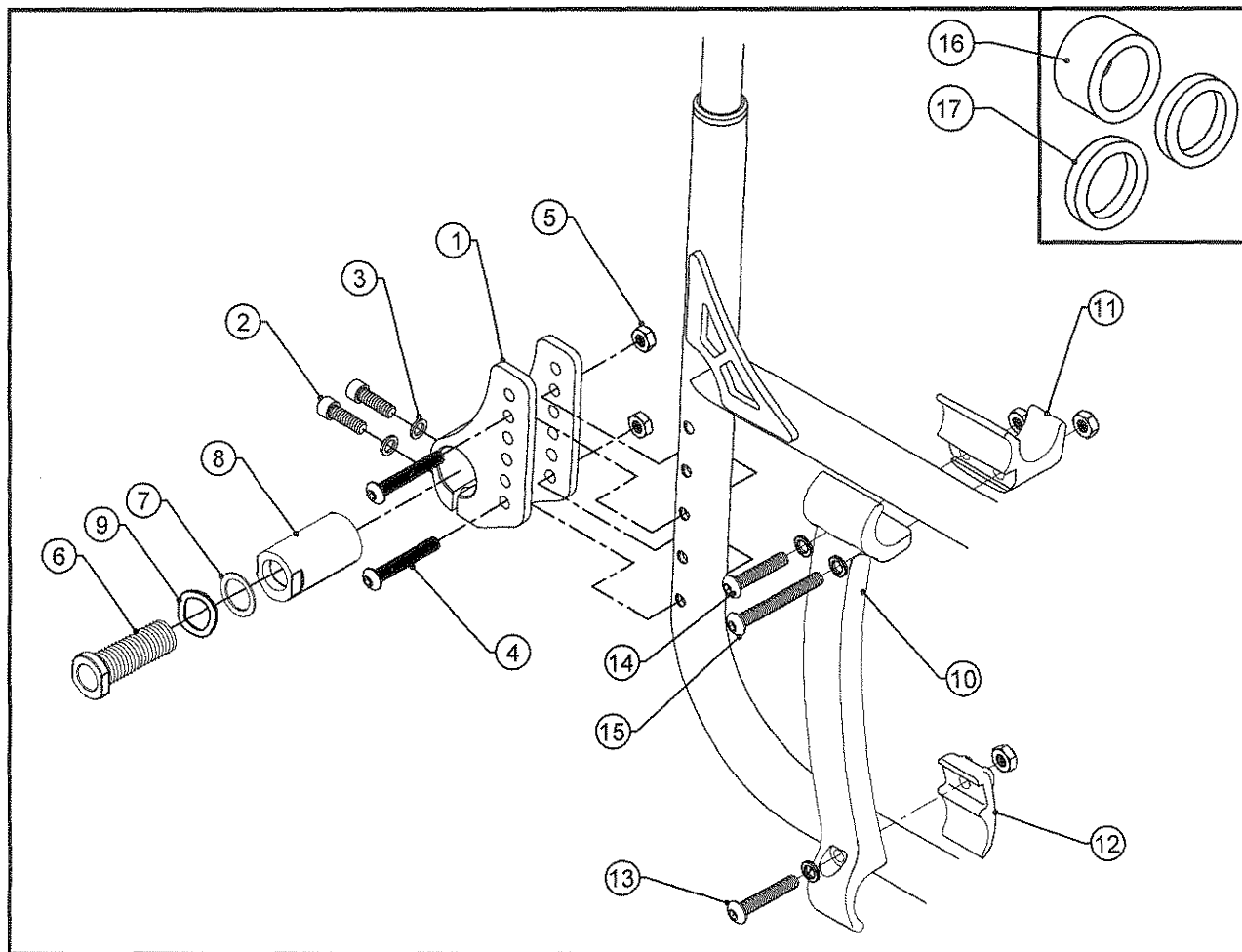


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP103102	AMPUTEE AXLE PLATE
2	4	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
3	10	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
4	4	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
5	10	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
6	1	CP100151	AXLE SLEEVE
7	1	HD100072	STEEL WASHER, .631 I.D X .868 O.D. X .36 THICK
8	1	TABLE ONE	CAMBER PLUG
9	1	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
10A	1	CP102754	DIE CAST AXLE PLATE RIGHT
10B	1	CP102755	DIE CAST AXLE PLATE LEFT
11	2	CP102575	VERTICAL AXLE PLATE COMPOSITE SADDLE CLAMP
12A	1	CP102756	DIE CAST TOE CLAMP RIGHT
12B	1	CP102757	DIE CAST TOE CLAMP LEFT
13	2	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
14	2	HD100437	1/4-28 X 1-3/8" BUTTON HEAD CAP SCREW, BLACK
15	2	HD100489	1/4-28 X 2-1/4" W/1-3/4" SHOULDER BHSCS, BLACK ZINC
16	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
17	2	HD100089	WASHER .906" X .638" X .125
1-5, 10-15		AS103581	AMPUTEE AXLE PLATE ASSEMBLY, PAIR
6-7, 9		AS116180	AXLE SLEEVE PACKAGE
6-9		TABLE ONE	FOLDER CAMBER ASSEMBLY
16-17		AS100945	REAR WHEEL SPACING PACKAGE

TABLE ONE - CAMBER PLUG

SEAT WIDTH	PART NUMBER	ASSEMBLY NUMBER
0°	CP100365	AS100301
2°	CP100364	AS100302
4°	CP100366	AS100303
6°	CP100867	AS100580
8°	CP100367	AS100304

AXLE ASSEMBLY

TiLite™ ADJUSTABLE DEPTH Aero X (Series 1 & 2),
ADJUSTABLE DEPTH 2GX (Series 1 & 2),

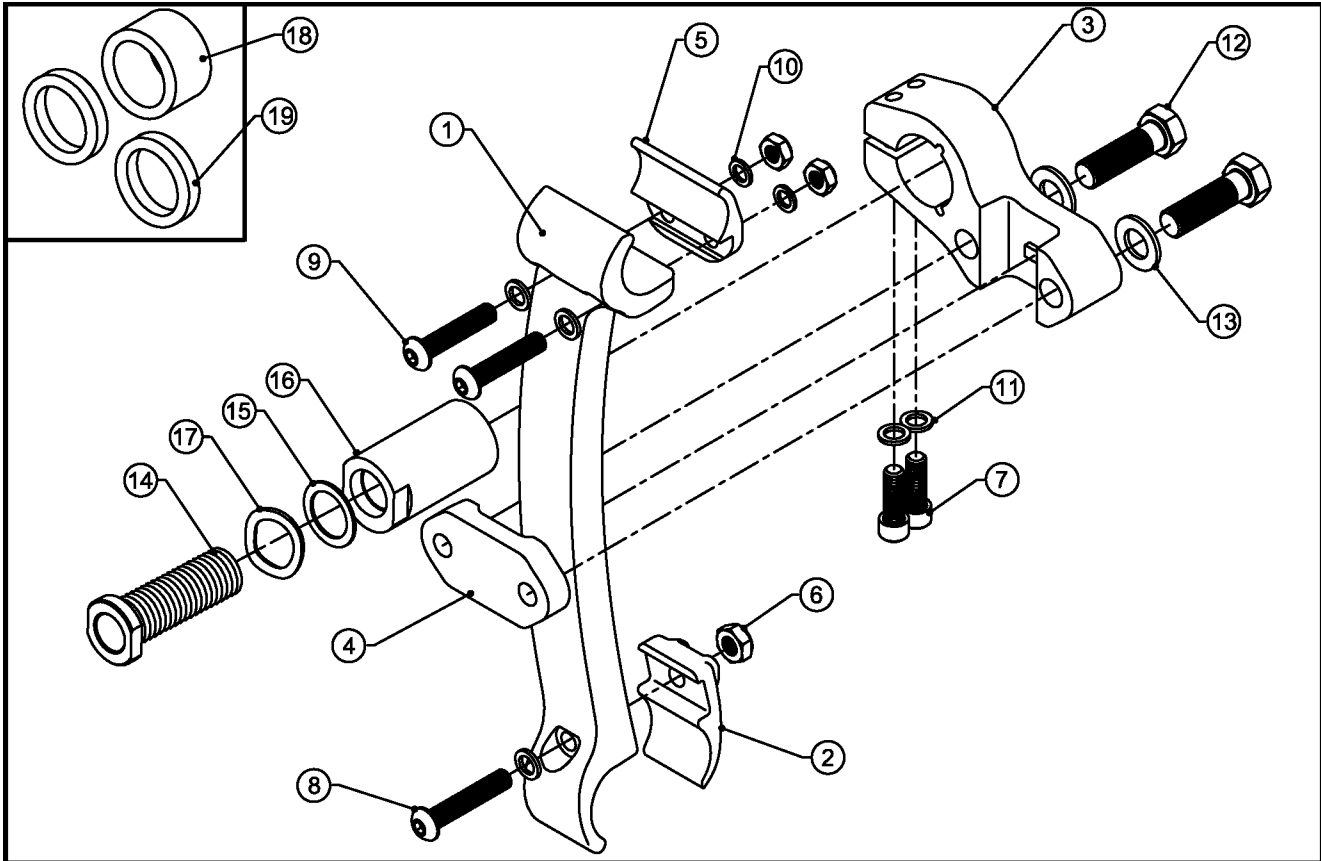


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP102754	DIE CAST AXLE PLATE RIGHT
1B	1	CP102755	DIE CAST AXLE PLATE LEFT
2A	1	CP102756	DIE CAST TOE CLAMP RIGHT
2B	1	CP102757	DIE CAST TOE CLAMP LEFT
3A	1	CP103198	AERO X CAMBER MOUNT RIGHT HAND
3B	1	CP103199	AERO X CAMBER MOUNT LEFT HAND
4	2	CP103200	AERO X / R CAMBER CLAMP
5	2	CP103130	RIGID AXLE PLATE CLAMP
6	6	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC PLATE
7	4	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
8	2	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
9	4	HD100369	1/4-28 X 1-1/2" BUTTON HEAD SOCKET CAP SCREW, BLACK ZINC
10	10	HD100056	1/4" AN WASHER .260 ID.500 UOD .0625 THICK, YELLOW ZINC PLATED
11	4	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	4	HD100470	3/8-16 X 1-1/2" GRADE 8 HEX HEAD CAP SCREW, BLACK ZINC
13	4	HD100353	3/8" AN WASHER STAINLESS STEEL
14	1	CP100151	AXLE SLEEVE
15	1	HD100072	STEEL WASHER, .631 I.D X .868 O.D. X .36 THICK
16	1	TABLE ONE	CAMBER PLUG
17	1	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
18	1	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
19	1	HD100089	WASHER .906" X .638" X .125
1-13		AS104833	DEPTH ADJ FRAME DIE CAST AXLE PLATE ASSEMBLY, PAIR
14-15, 17		AS116180	AXLE SLEEVE PACKAGE
14-17		TABLE ONE	FOLDER CAMBER ASSEMBLY
18-19		AS100945	REAR WHEEL SPACING PACKAGE

TABLE ONE - CAMBER PLUG

SEAT WIDTH	PART NUMBER	ASSEMBLY NUMBER
0°	CP100365	AS100301
2°	CP100364	AS100302
4°	CP100366	AS100303
6°	CP100867	AS100580
8°	CP100367	AS100304

AXLE ASSEMBLY

TiLite™ X, SX

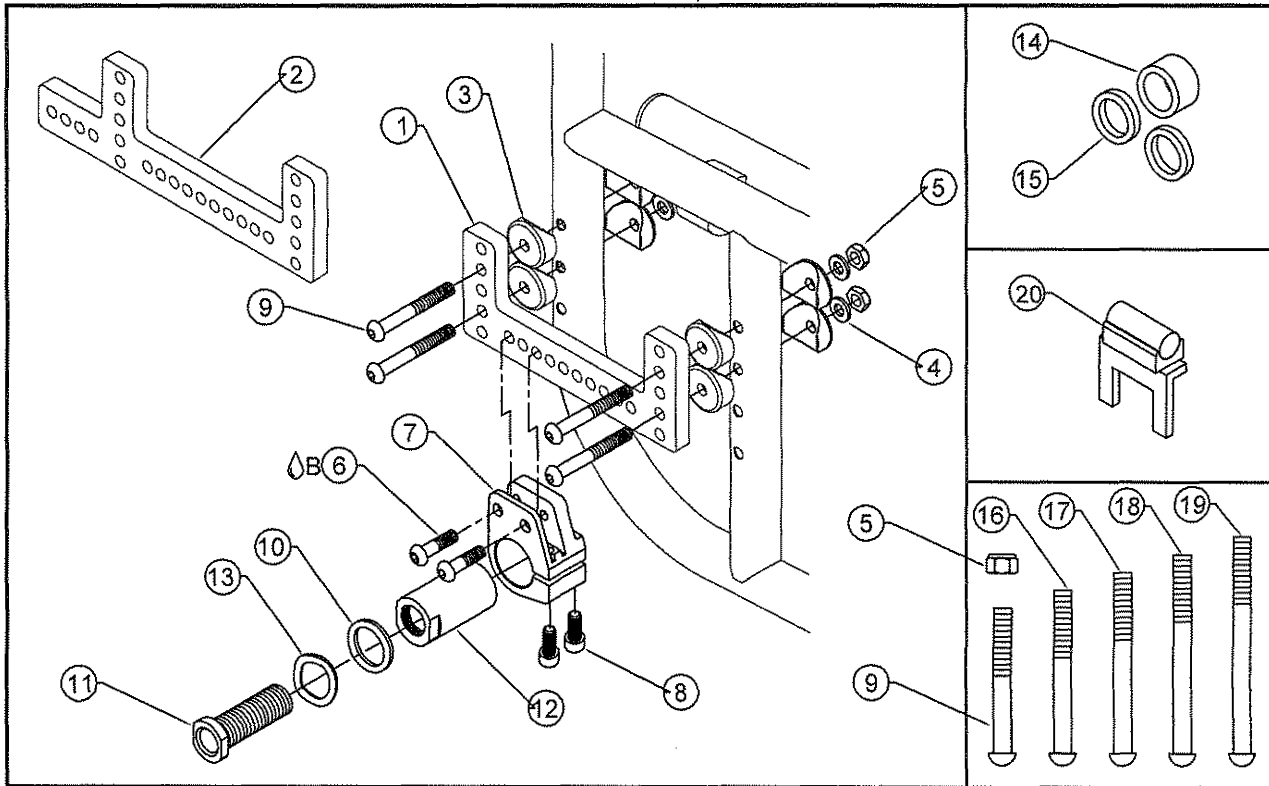

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP100532	FOLDING CHAIR AXLE PLATE EXTENDED
2	2	CP100690	AMPUTEE AXLE PLATE FOLDER EXTENDED
3	16	HD100014	COVERED WASHER 1" O.D. X 1/4" I.D. X 1/8" THK
4	8	HD100022	1/4 SAE FLAT WASHER
5	8	HD100021	1/8-28 X .218 NYLOCK LOCKNUT
6A	4	HD100065	1/4-20 X 1" BUTTON HEAD CAP SCREW, CLEAR ZINC PLATED
7A	1	CP100536	CAMBER TUBE MOUNT RIGHT FOLDER
7B	1	CP100537	CAMBER TUBE MOUNT LEFT FOLDER
8	4	HD100008	1/4-20 X 5/8" SOCKET HEAD CAP SCREW
9	8	HD100104	1/4-28 X 2.00" BUTTON HEAD CAP SCREW
10	2	HD100072	STEEL WASHER, .631 I.D X .868 O.D. X .36 THICK
11	2	CP100151	AXLE SLEEVE
12	2	TABLE ONE	0°, 2°, 4°, 6°, OR 8° CAMBER PLUG
13	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
14	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
15	4	HD100089	WASHER .906" X .638" X .125
16	8	HD100105	1/4-28 X 2-1/4" BUTTON HEAD CAP SCREW
17	8	HD100245	1/4-28 X 2-1/2" BUTTON HEAD CAP SCREW
18	8	HD100351	1/4-28 X 2-3/4" BUTTON HEAD CAP SCREW
19	8	HD100598	1/4-28 X 3" BUTTON HEAD CAP SCREW
20	1	CP101984	BUBBLE LEVEL BRACKET FOR ADJUSTABLE CAMBER
14-15		AS100945	REAR WHEEL SPACING PACKAGE
5,9,16-19		AS102774	ACCESSORY BOLT PACKAGE
1,3-9		AS100276	SX/X AXLE PLATE ASSY, PAIR
2-9		AS100640	SX/X AMPUTEE AXLE PLATE ASSY, PAIR
10-13		TABLE TWO	FOLDER CAMBER ASSY
10-11, 13		AS116180	AXLE SLEEVE PACKAGE
TABLE ONE - CAMBER PLUG			TABLE TWO - CAMBER ASSY


CAMBER DEGREE	PART NUMBER
0°	CP100365
2°	CP100364
4°	CP100366
6°	CP100867
8°	CP100367
2° EXT	CP100849
4° EXT	CP100850

CAMBER DEGREE	ASSY NUMBER
0°	AS100301
2°	AS100302
4°	AS100303
6°	AS100580
8°	AS100304
2° EXT	AS104154
4° EXT	AS104155



ADJUSTABLE AXLE ASSY

TiLite™ TX

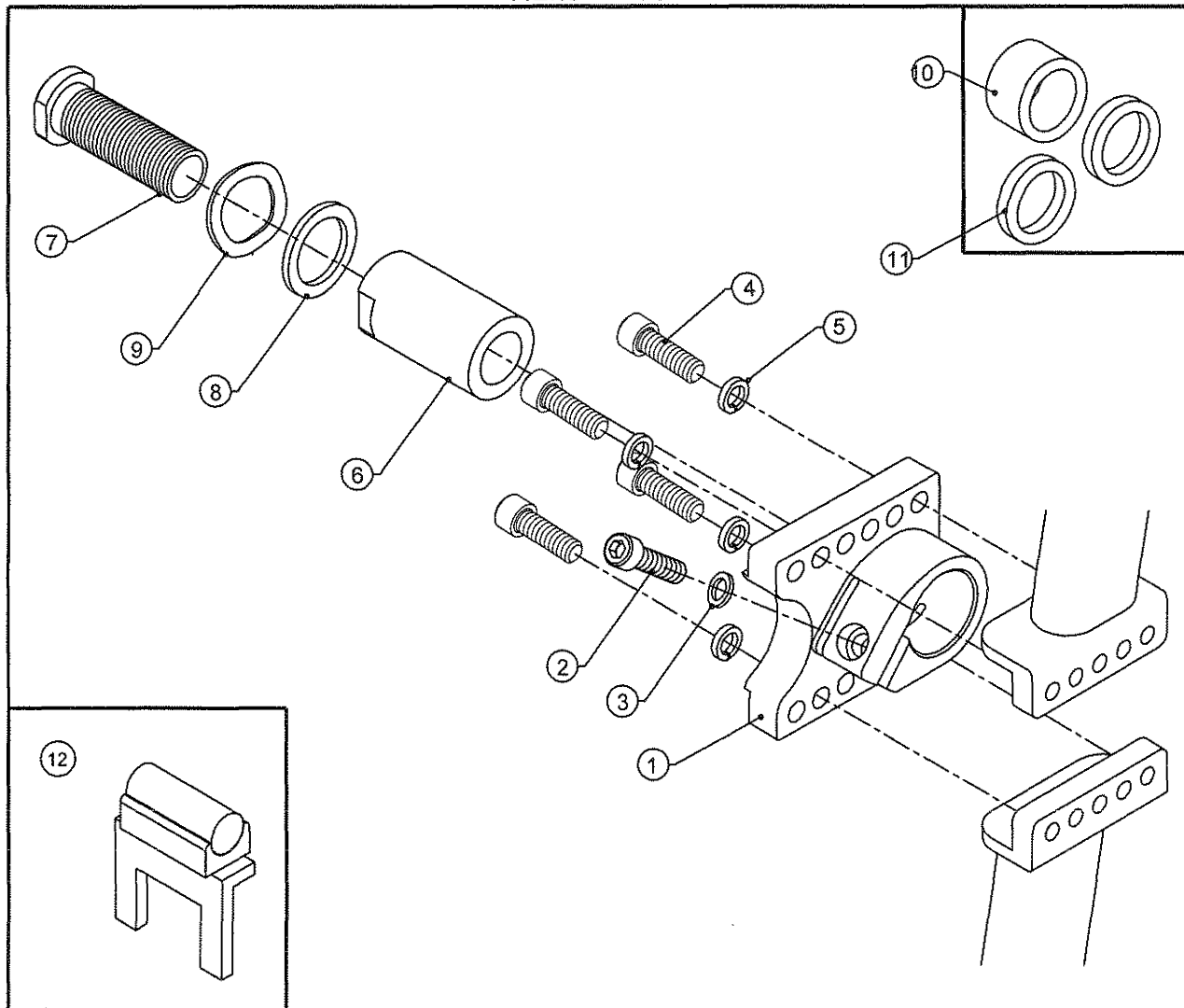

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP102542	ADJUSTABLE TX CAMBER CLAMP
2	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
3	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
4	8	HD100436	1/4-28 X 1" SOCKET HEAD CAP SCREW, BLACK
5	8	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
6	2	TABLE ONE	0°, 2°, OR 4° FOLDING CHAIR CAMBER PLUG
7	2	CP100151	AXLE SLEEVE
8	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .36 THICK
9	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
10	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
11	4	HD100089	WASHER .906" X .638" X .125
12	1	CP101984	BUBBLE LEVEL BRACKET FOR ADJUSTABLE CAMBER
1-5		AS102982	TX ADJUSTABLE AXLE ASSEMBLY, PAIR
10-11		AS100945	REAR WHEEL SPACING PACKAGE
6-9		TABLE TWO	FOLDER CAMBER ASSY
7-9		AS116180	AXLE SLEEVE PACKAGE

**TABLE ONE
CAMBER PLUG**

°CAMBER	PART NO.
0°	CP100365
2°	CP100364
4°	CP100366


**TABLE TWO
FOLDER CAMBER ASSY**

°CAMBER	ASSY NO.
0°	AS100301
2°	AS100302
4°	AS100303



AXLE ASSY

TiLite™ TX

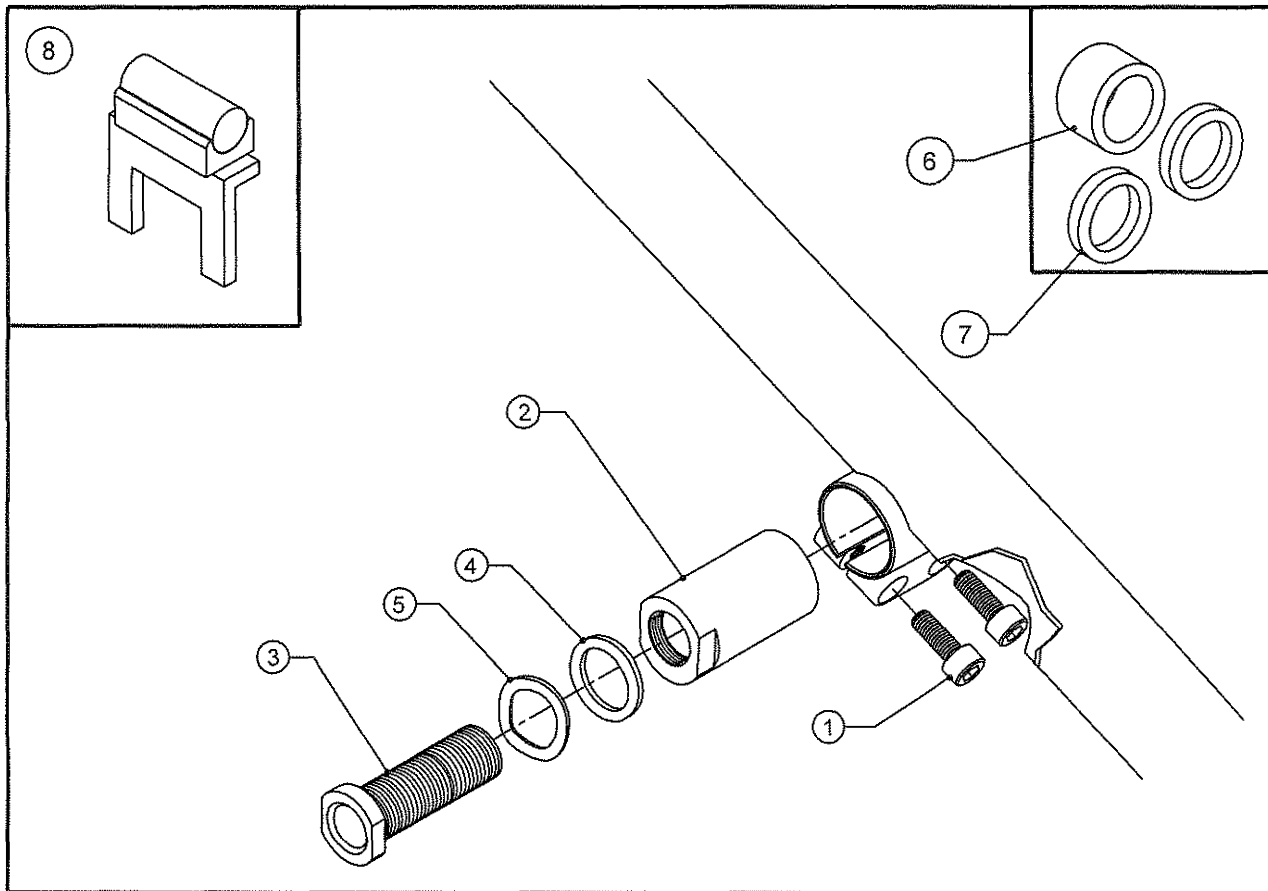

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	4	HD100008	1/4-20 X 5/8 LONG SOCKET HEAD CAP SCREW
2	2	TABLE ONE	0°, 2°, OR 4° FOLDING CHAIR CAMBER PLUG
3	2	CP100151	AXLE SLEEVE
4	2	HD100072	STEEL WASHER, .631 I.D. X .868 O.D. X .36 THK
5	2	HD100511	WAVE SPRING WASHER .645 ID X .870 OD X .062 THK
6	2	CP100962	SPINERGY WHEEL AXLE SLEEVE SPACER
7	4	HD100089	WASHER .906\" X .638\" X .125
8	1	CP101984	BUBBLE LEVEL BRACKET FOR ADJUSTABLE CAMBER
6-7		AS100945	REAR WHEEL SPACING PACKAGE
2-5		TABLE TWO	FOLDER CAMBER ASSY
3-5		AS116180	AXLE SLEEVE PACKAGE



TABLE ONE

°CAMBER	PART NO.
0°	CP100365
2°	CP100364
4°	CP100366

TABLE TWO

°CAMBER	ASSY NO.
0°	AS100301
2°	AS100302
4°	AS100303

QUICK RELEASE AXLES

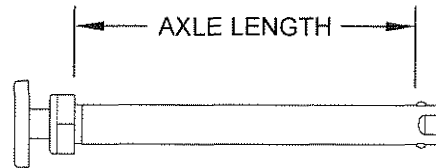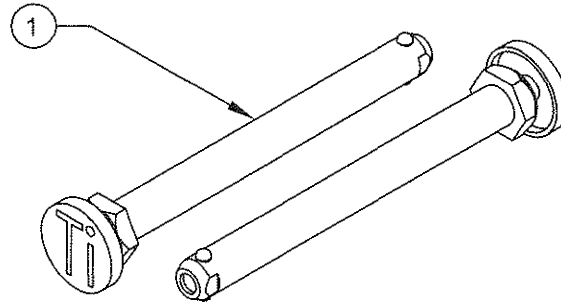

FOR REFERENCE


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100932	QUICK RELEASE AXLE WITH LOGO 4.55" (4.45" MIN)
1B	1	CP100141	QUICK RELEASE AXLE WITH LOGO 4.37" (4.20" MIN)
1C	1	CP101035	QUICK RELEASE WITH LOGO 4.00" (3.68" MIN)
1D	1	CP102619	QUICK RELEASE AXLE WITH LOGO 4.00" (3.88" MIN)
1E	1	CP107396	TI QUICK RELEASE WITH LOGO 4.305" (4.22" MIN)
1F	1	CP107397	TI QUICK RELEASE WITH LOGO 4.00" (3.72" MIN)

TABLE ONE

AXLE	WHEELS
CP100932	HIGH FLANGE
CP100141	26" SPINERGY (SPOX AND LX), TOPOLINO
CP101035	22"-25" SPINERGY (SPOX AND LX)
CP102619	SHADOW, GOLZ, MAG WHEEL
CP107396	26" SPINERGY (SPOX AND LX), TOPOLINO
CP107397	22"-25" SPINERGY (SPOX AND LX), SHADOW, GOLZ, MAG WHEEL

QUAD QUICK RELEASE AXLE


FITS HIGH FLANGE HUBS AND LOW FLANGE HUBS

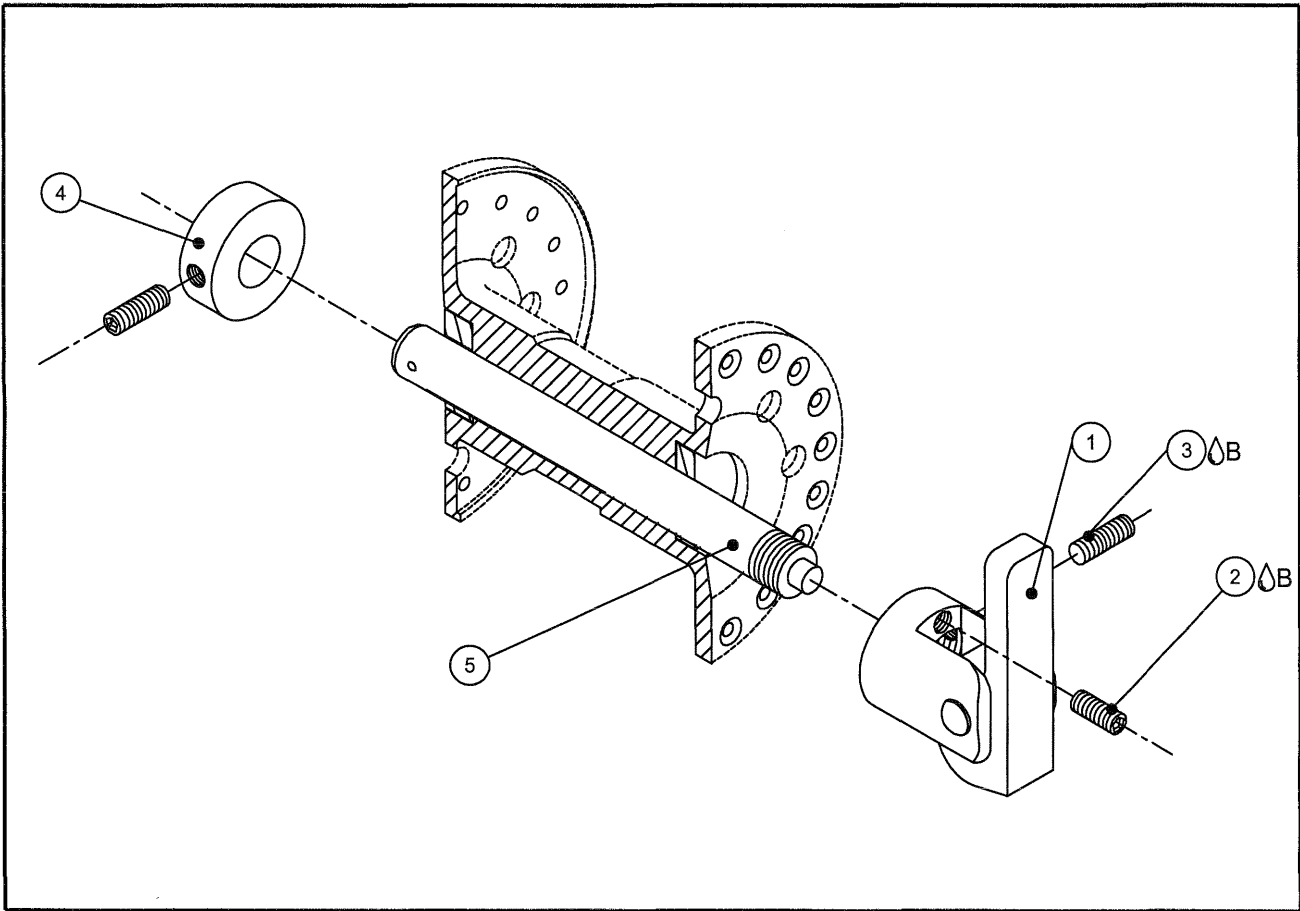

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	AS100726	QUADRAPLEGIC NUT ASSY
2 Δ B	1	HD100131	8-32 X 3/8" SOCKET SET SCREW, DOG POINT
3 Δ B	2	HD100132	8-32 X 3/16" SOCKET SET SCREW, NYLON TIP
4	1	CP100757	COLLAR, QUADRAPLEGIC QUICK RELEASE AXLE
5	1	CP100758	STEM, QUADRAPLEGIC QUICK RELEASE AXLE
1-5		AS100334	QUADRAPLEGIC QUICK RELEASE AXLE


QUAD QUICK RELEASE AXLE

FITS SPINERGY LITE HUBS

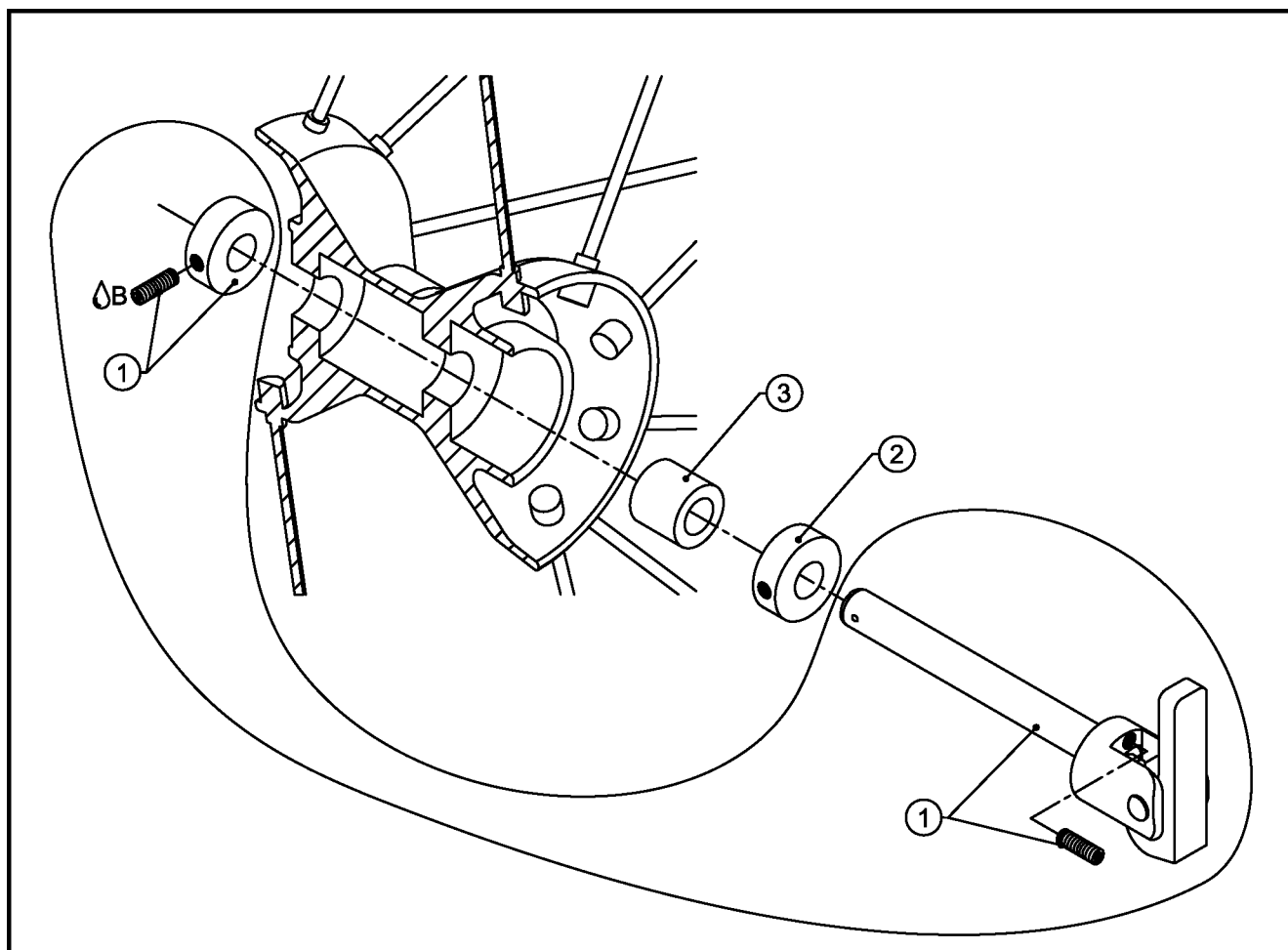

ITEM	QUANTITY	PART NO.	DESCRIPTION
1B	1	AS100334	QUAD QUICK RELEASE AXLE W/ SETSCREW
2	1	CP100757	COLLAR, QUADRAPLEGIC QUICK RELEASE AXLE
3	1	CP100346	CASTER EXTENSION 1/2"
1-3		AS102908	QUAD QUICK RELEASE AXLE FOR SPINERGY SPOX/LX LITE HUB


FITS SPINERGY WIRE WHEELS

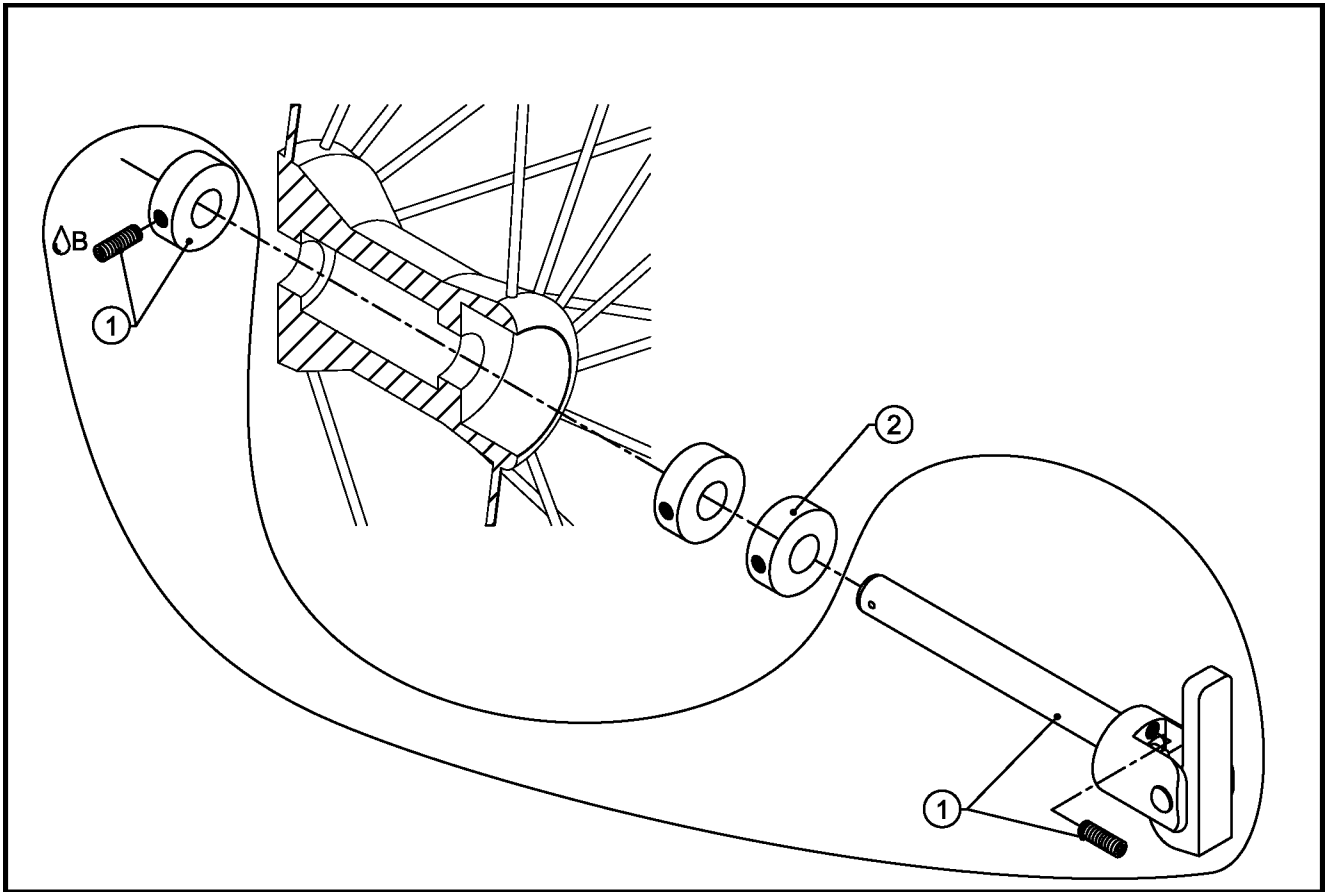

ITEM	QUANTITY	PART NO.	DESCRIPTION
10B	1	AS100334	QUAD QUICK RELEASE AXLE W/ SETSCREW
2	2	CP100757	COLLAR, QUADRAPLEGIC QUICK RELEASE AXLE
1-2		AS103435	QUAD QUICK RELEASE AXLE FOR SPINERGY WIRE HUB

QUAD QUICK RELEASE AXLE


FITS TOPOLINO WHEELS

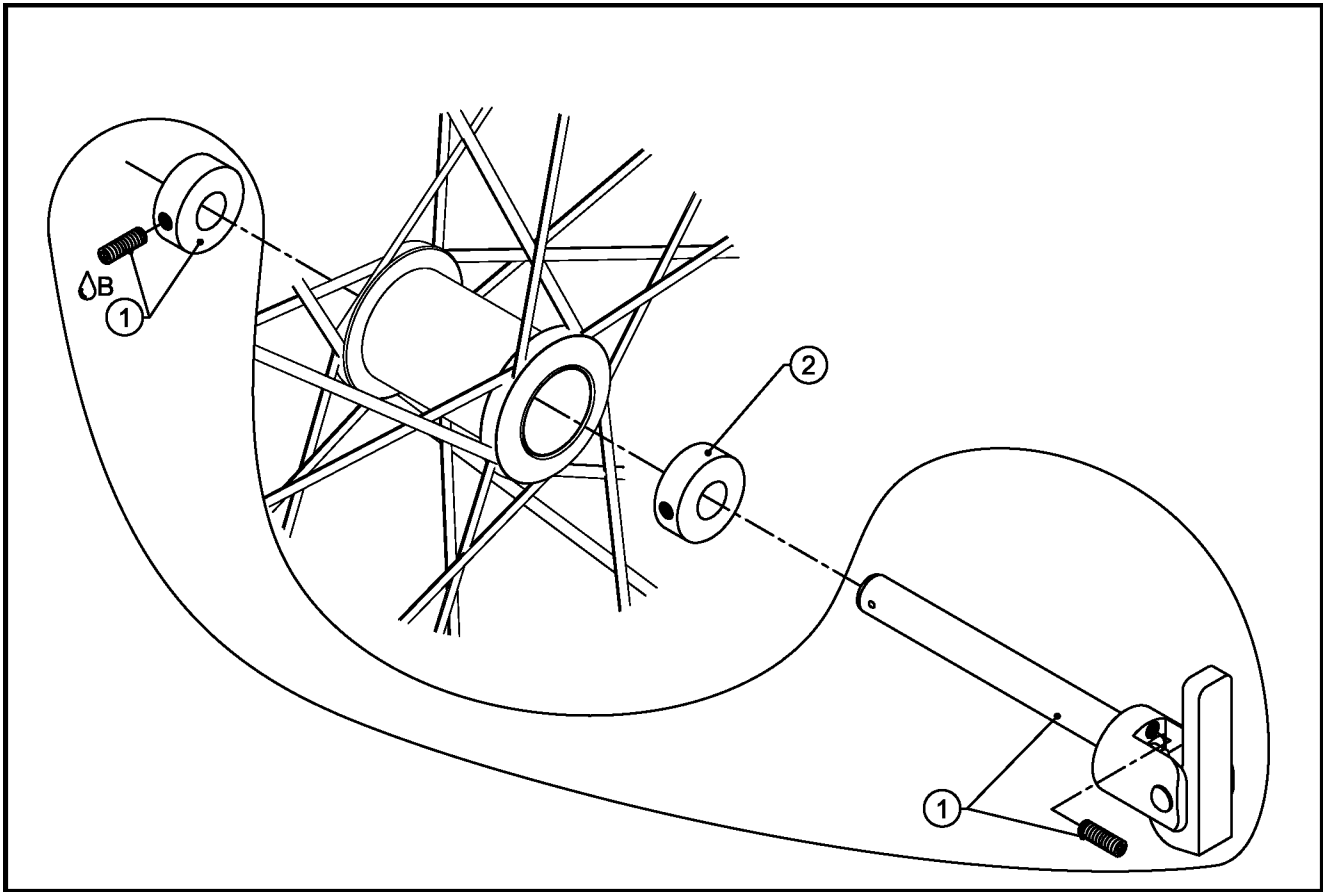

ITEM	QUANTITY	PART NO.	DESCRIPTION
1 B	1	AS100334	QUAD QUICK RELEASE AXLE W/ SETSCREW
2	2	CP100757	COLLAR, QUADRAPLEGIC QUICK RELEASE AXLE
1-2		AS116026	QUAD QUICK RELEASE AXLE FOR TOPOLINO HUB

QUAD QUICK RELEASE AXLE


FITS SPINERGY CARBON BLADE WHEELS

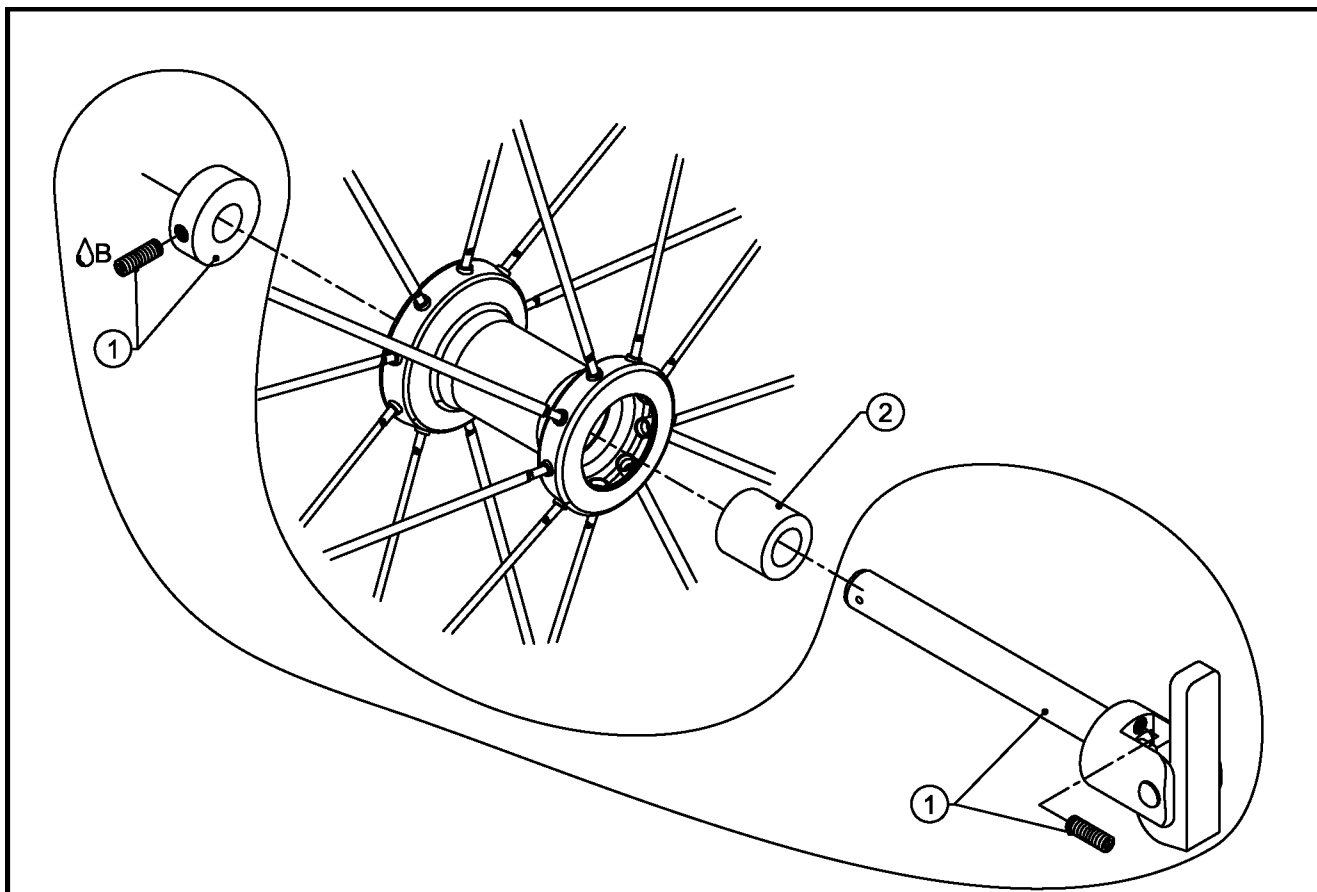

ITEM	QUANTITY	PART NO.	DESCRIPTION
1B	1	AS100334	QUAD QUICK RELEASE AXLE W/ SETSCREW
2	1	CP100346	CASTER EXTENSION 1/2"
1-2		AS118849	QUAD QUICK RELEASE AXLE FOR CARBON BLADE HUB

SPINERGY 22 & 24" - POWER AXLE

ALL TiLite™ REINFORCED FRAMES (REQUIRED FOR POWER ADD-ON SYSTEMS)

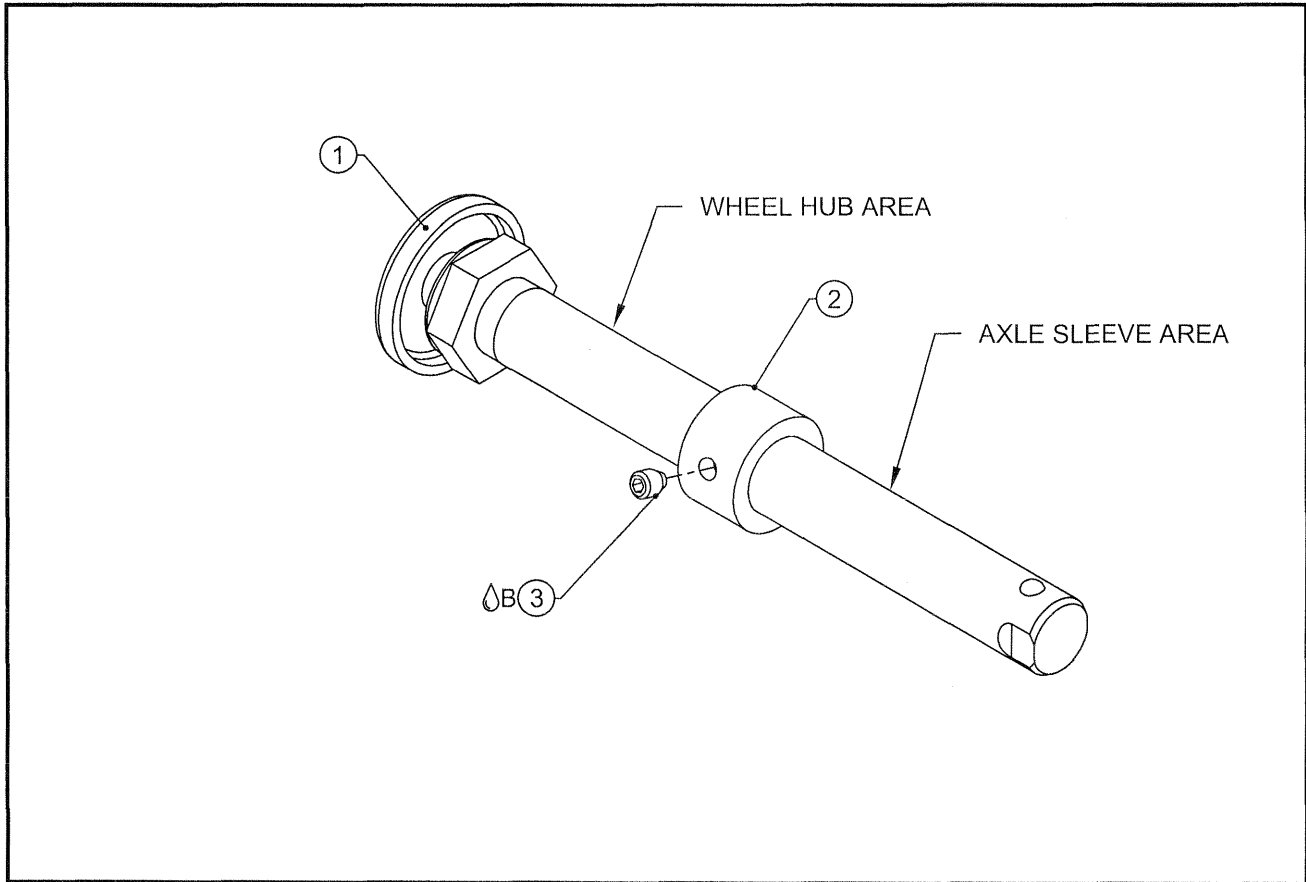

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100141	STANDARD AXLE FOR 26" WHEEL
2	1	CP102404	MANUAL - POWER SPACER
3	1	HD100132	8-32 SET SCREW WITH NYLON TIP
1-3		AS102893	SPINERGY 22 & 24" - POWER AXLE
1-3		AS102894	SPINERGY 22 & 24" - POWER AXLE (PAIR)