

Permobil

# Coloured lights package

The coloured lights package is now available on all Power Platform enabled wheelchairs. If you need assistance updating your wheelchair for the coloured lights package, please contact your local Permobil Territory Sales Manager or our Customer Service team at 0800 115 222.

## Requires at least firmware 1.57 and a configuration package for the coloured lights.

Once the packages are sent to your chair, you may see notifications on the display indicating steps of the installation process.


### MyPermobil app:

1. Install or Update MyPermobil app to the latest version (must be at least Version 1.75).
2. Complete steps to create an account and activate your chair (if not previously done). See video [here](#).
3. Ensure chair is in BT pairing mode. See video [here](#).
4. Once chair is activated and connected to the MyPermobil app, go to the *Settings* tab.
5. Select *My Wheelchair* from the menu.
6. Select *Wheelchair Settings*.
7. Select *Coloured Lights*.
8. Toggle coloured lights ON (*This button will turn coloured lights off and back on without requiring the joystick or an external switch*).
9. Use sliders to select Colour, Colour intensity, & Brightness.
10. Choose *Set Colour* and the lights will change to the colour selected within 2-3 seconds.


**Note:** The switch for the lights will function as follows when coloured lights are programmed.

- Press #1 - Normal lights
- Press #2 - Coloured lights
- Press #3 - Lights off


Step 4

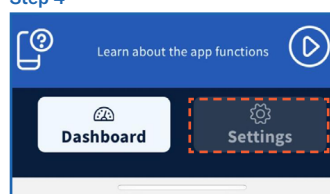


Step 6

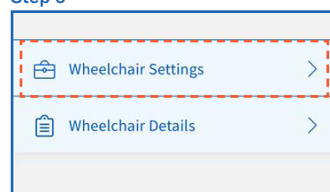


Step 8

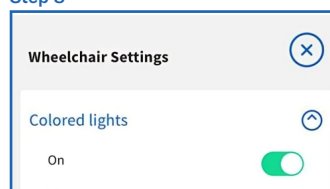

Step 5

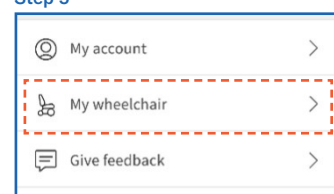

Step 7

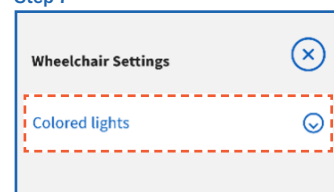

Step 9 &amp; 10

