


# MyPermobil—Activation Process




**NOTE:** WiFi and email access is required for set-up


## Getting started

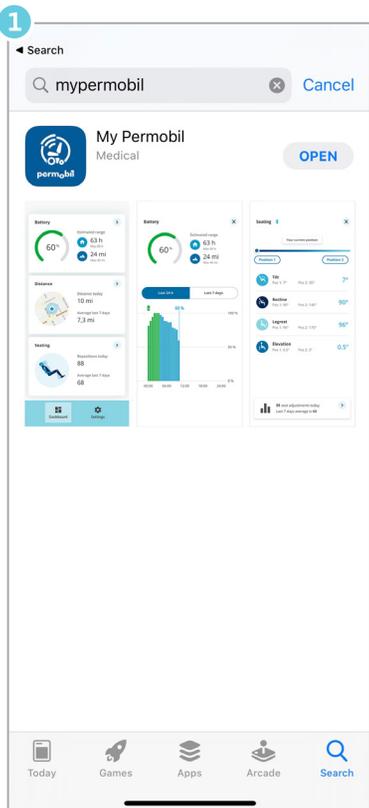


Download *MyPermobil* on your Android or Apple phone

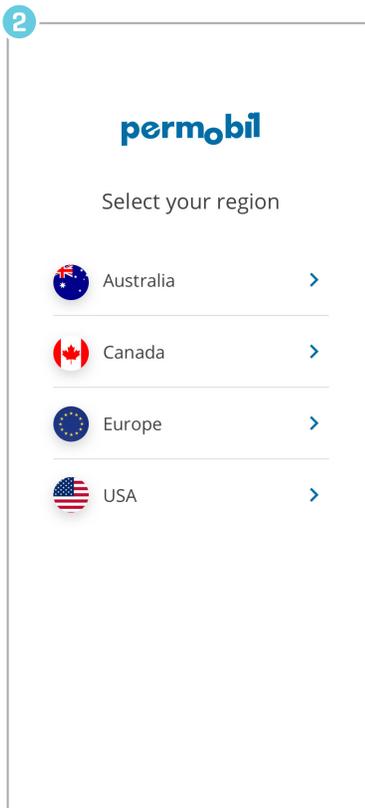

Choose your region

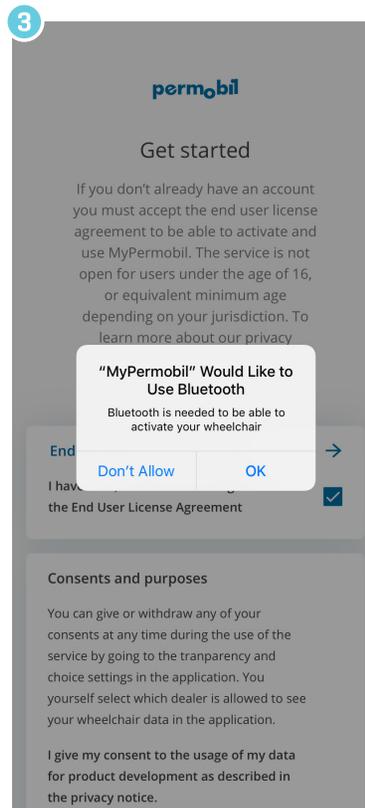

Accept to allow Bluetooth™

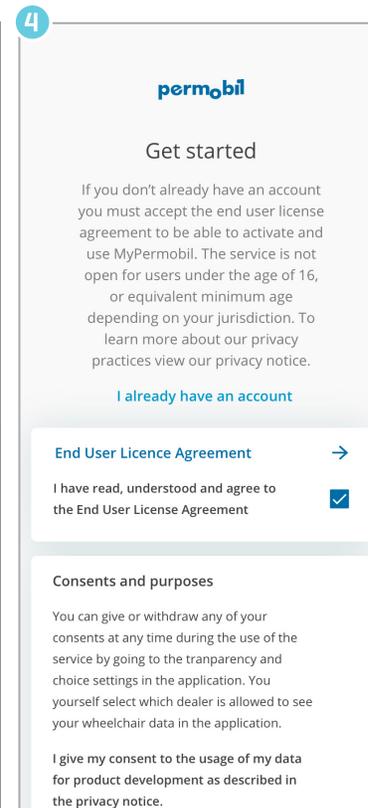

Read/accept MyPermobil policy




MyPermobil




# MyPermobil—Activation Process

**NOTE:** WiFi and email access is required for set-up


## Create account

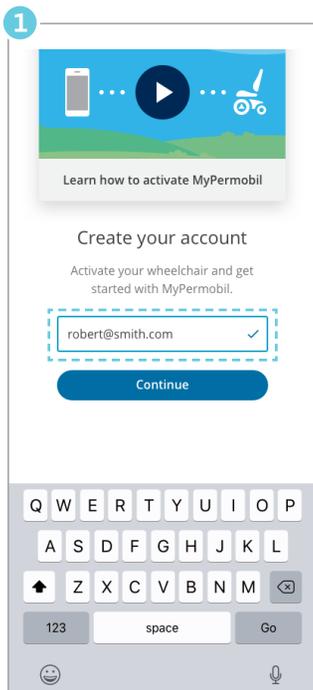

Enter e-mail address

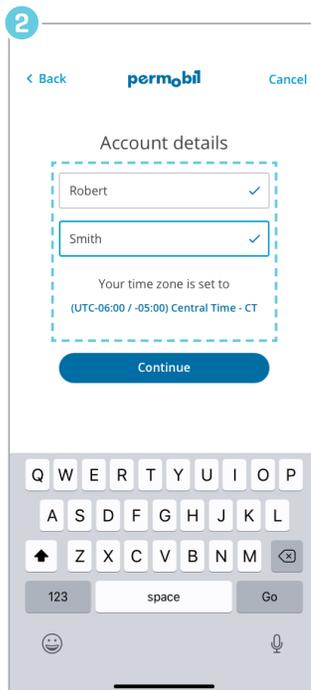


Enter your name and set time zone

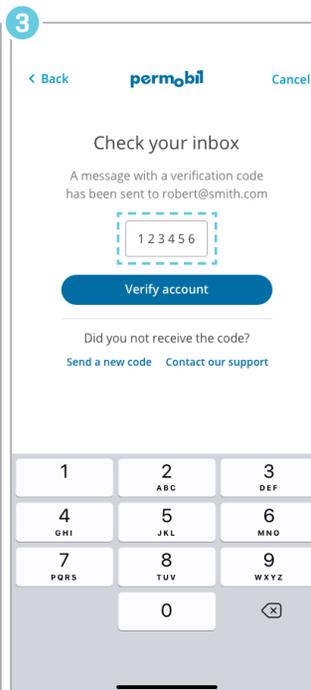


Find code in email, enter it and hit "Verify account"

## Activate

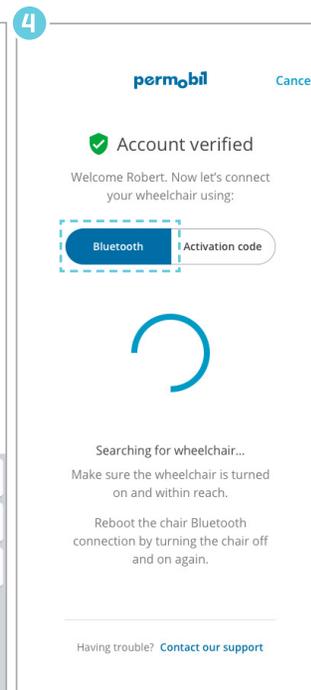


Select Bluetooth and turn on your wheelchair

*If your wheelchair is already on, turn it off and on again in order to reboot Bluetooth connection*

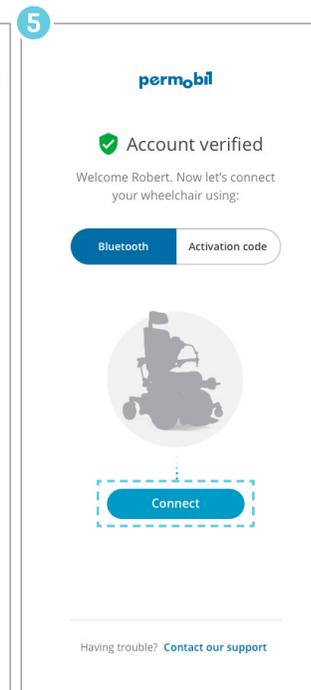

Click on the Connect button

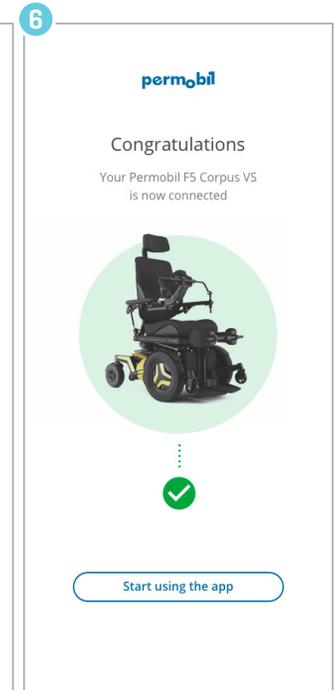

You are connected!