


Casters, Forks and Caster Caps

Section 8



WIDE BEARING FORK ASSEMBLY

TiLite™ ZR, TR(SERIES 2), YR(SERIES 2), TX

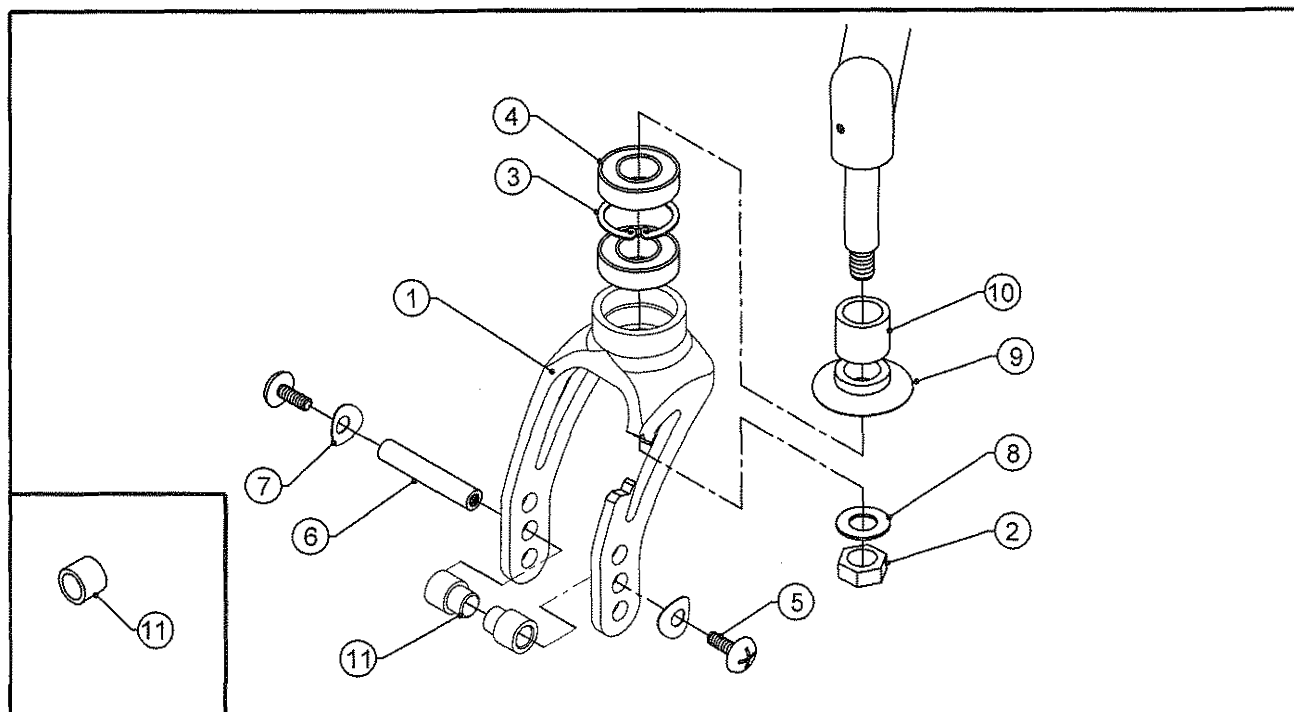

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP106957	BEARING FORK FOR 1.5" WIDE CASTER
1B	1	CP106956	BEARING FORK FOR 1.5" WIDE CASTER SHORT
2	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
3	1	HD100330	1-1/8" INTERNAL RETAINING RING
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
6	1	CP103041	CASTER AXLE LONG
7	2	CP104375	C SPRING WASHER
8	1	HD100353	3/8" AN WASHER STAINLESS STEEL
9	1	CP102048	CASTER CAP BEARING FORK
10A	1	CP100346	CASTER EXTENSION 1/2"
10B	1	CP100347	CASTER EXTENSION 1"
5,6,7		AS116086	CASTER EXTENSION 1"
11	2	TABLE ONE	CASTER SPACER
1A-9,11		TABLE ONE	WIDE BEARING FORK ASSEMBLY
1A-9,10A,11		TABLE ONE	WIDE BEARING FORK ASSEMBLY W/ 1/2" EXT
1A-9,10B,11		TABLE ONE	WIDE BEARING FORK ASSEMBLY W/ 1" EXT
1B-9,11		TABLE ONE	WIDE SHORT BEARING FORK SHORT ASSEMBLY
1B-9,10A,11		TABLE ONE	WIDE SHORT BEARING FORK ASSEMBLY W/ 1/2" EXT
1B-9,10B,11		TABLE ONE	WIDE SHORT BEARING FORK ASSEMBLY W/ 1" EXT

ASSEMBLIES ON NEXT PAGE

WIDE BEARING FORK ASSEMBLY

TiLite™ ZR, TR(SERIES 2), YR(SERIES 2), TX


TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK	STD FORK 1/2" EXT	STD FORK 1" EXT	SHORT FORK	SHORT FORK 1/2" EXT	SHORT FORK 1" EXT
3" MICRO	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297
3" VOLCANIC	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297
4" PLASTIC W/ POLY	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297
4" VOLCANIC	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297
4" LITESPEED W/ POLY	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
5" PLASTIC W/ POLY	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297
5" LITESPEED W/ POLY	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
6" PLASTIC W/ POLY	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297
6" LITESPEED W/ POLY	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111348	AS111349	AS111350	AS111351	AS111352	AS111353
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS104265	AS104267	AS104268	AS104295	AS104296	AS104297



WIDE BEARING FORK ASSEMBLY

TiLite™ ZRA

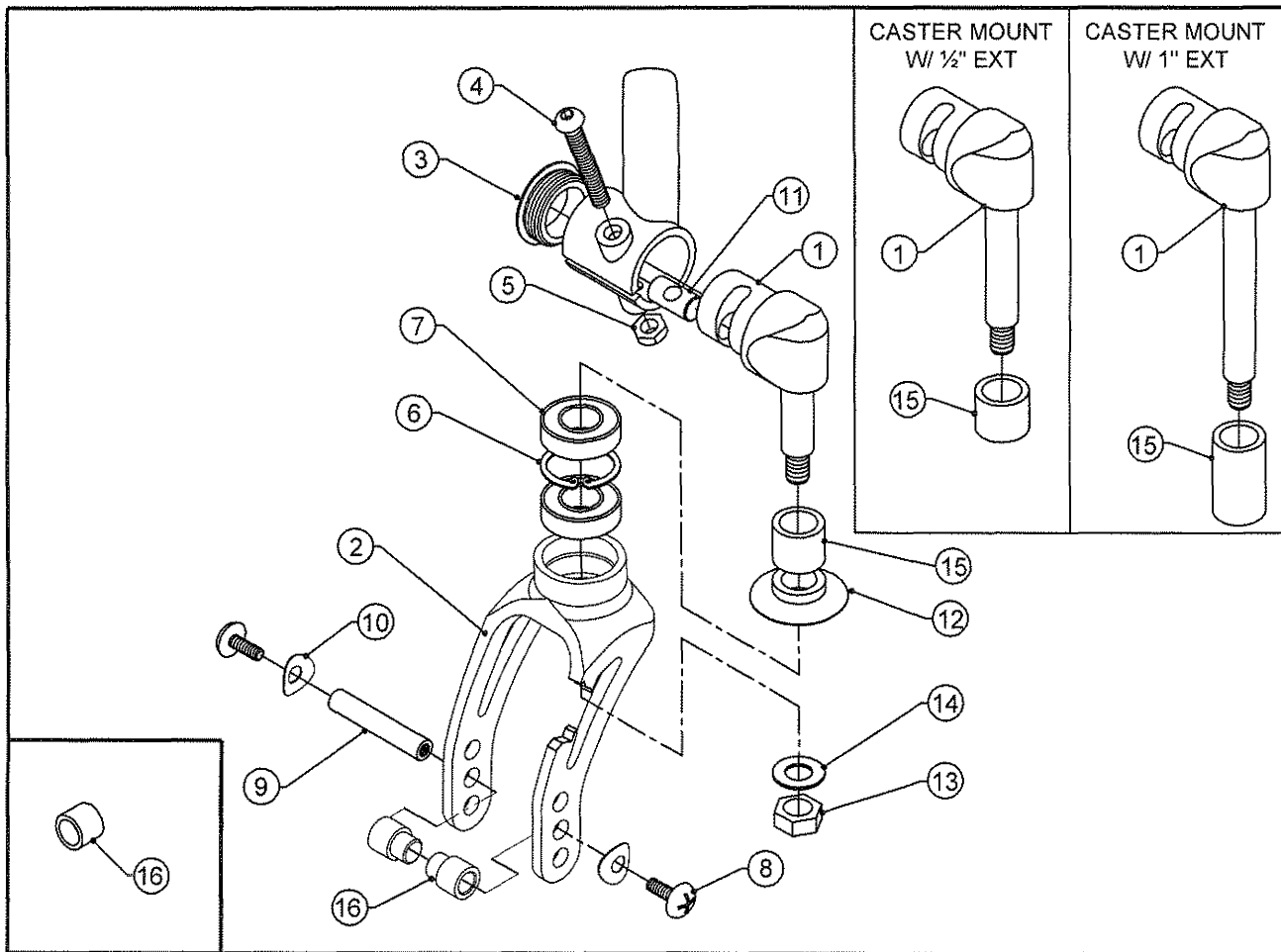

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS102842	CASTER MOUNT FOR BEARING FORK RIGHT ASSEMBLY
1B	1	AS102843	CASTER MOUNT FOR BEARING FORK LEFT ASSEMBLY
1C	1	AS102844	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT RIGHT ASSEMBLY
1D	1	AS102845	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT LEFT ASSEMBLY
1E	1	AS102846	CASTER MOUNT FOR BEARING FORK W/ 1" EXT RIGHT ASSEMBLY
1F	1	AS102847	CASTER MOUNT FOR BEARING FORK W/ 1" EXT LEFT ASSEMBLY
2A	1	CP106957	BEARING FORK FOR 1.5" WIDE CASTER
2B	1	CP106956	BEARING FORK FOR 1.5" WIDE CASTER SHORT
3	1	CP102064	ENDCAP 1.25 OD X .125 WALL
4	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
5	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
6	1	HD100330	1-1/8" INTERNAL RETAINING RING
7	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
8	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
9	1	CP103041	CASTER AXLE LONG
10	2	CP104375	C SPRING WASHER
11	1	CP102142	CASTER MOUNT DOWEL Ø.437
12	1	CP102048	CASTER CAP BEARING FORK
13	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
14	1	HD100353	3/8" AN WASHER STAINLESS STEEL
15A	1	CP100346	CASTER EXTENSION 1/2"
15B	1	CP100347	CASTER EXTENSION 1"
8,9,10		AS116086	HARDWARE KIT FOR BEARING FORK ASSY.
16	2	TABLE ONE	CASTER SPACER

ASSEMBLIES ON NEXT PAGE

WIDE BEARING FORK ASSEMBLY (CONT.)

TiLite™ ZRA

ITEM	QTY	PART NO.	DESCRIPTION
1B,2A-14,16	TABLE ONE	ZRA WIDE BEARING FORK ASSEMBLY LEFT	
1A,2A-14,16	TABLE ONE	ZRA WIDE BEARING FORK ASSEMBLY RIGHT	
1D,2A-15A,16	TABLE ONE	ZRA WIDE BEARING FORK ASSEMBLY LEFT W/ 1/2" EXT	
1C,2A-15A,16	TABLE ONE	ZRA WIDE BEARING FORK ASSEMBLY RIGHT W/ 1/2" EXT	
1F,2A-15B,16	TABLE ONE	ZRA WIDE BEARING FORK ASSEMBLY LEFT W/ 1" EXT	
1E,2A-15B,16	TABLE ONE	ZRA WIDE BEARING FORK ASSEMBLY RIGHT W/ 1" EXT	
1F,2B-14,16	TABLE TWO	ZRA WIDE SHORT BEARING FORK ASSEMBLY LEFT	
1E,2B-14,16	TABLE TWO	ZRA WIDE SHORT BEARING FORK ASSEMBLY RIGHT	
1D,2B-15A,16	TABLE TWO	ZRA WIDE SHORT BEARING FORK ASSEMBLY LEFT W/ 1/2" EXT	
1C,2B-15A,16	TABLE TWO	ZRA WIDE SHORT BEARING FORK ASSEMBLY RIGHT W/ 1/2" EXT	
1F,2B-15B,16	TABLE TWO	ZRA WIDE SHORT BEARING FORK ASSEMBLY LEFT W/ 1" EXT	
1E,2B-15B,16	TABLE TWO	ZRA WIDE SHORT BEARING FORK ASSEMBLY RIGHT W/ 1" EXT	

TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK LEFT	STD FORK RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1" EXT LEFT	STD FORK 1" EXT RIGHT
3" MICRO	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260
3" VOLCANIC	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260
4" PLASTIC W/ POLY	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260
4" VOLCANIC	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260
4" LITESPEED W/ POLY	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
5" PLASTIC W/ POLY	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260
5" LITESPEED W/ POLY	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
6" PLASTIC W/ POLY	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260
6" LITESPEED W/ POLY	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111354	AS111355	AS111356	AS111357	AS111358	AS111359
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS104253	AS104254	AS104257	AS104258	AS104259	AS104260


TABLE TWO - SPACERS AND SHORT FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	SHORT FORK LEFT	SHORT FORK RIGHT	SHORT FORK 1/2" EXT LEFT	SHORT FORK 1/2" EXT RIGHT	SHORT FORK 1" EXT LEFT	SHORT FORK 1" EXT RIGHT
3" MICRO	CP103945	CP103945	AS104289	AS104290	AS104291	AS104292	AS104293	AS104294
3" VOLCANIC	CP103945	CP103945	AS104289	AS104290	AS104291	AS104292	AS104293	AS104294
4" PLASTIC W/ POLY	CP103945	CP103945	AS104289	AS104290	AS104291	AS104292	AS104293	AS104294
4" VOLCANIC	CP103945	CP103945	AS104289	AS104290	AS104291	AS104292	AS104293	AS104294
4" LITESPEED W/ POLY	CP104405	CP104406	AS111360	AS111361	AS111362	AS111363	AS111364	AS111365
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111360	AS111361	AS111362	AS111363	AS111364	AS111365
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP103945	CP103945	AS104289	AS104290	AS104291	AS104292	AS104293	AS104294



MONO FORK ASSEMBLY

TiLite™ ZRA (SERIES 2), TRA (SERIES 2)

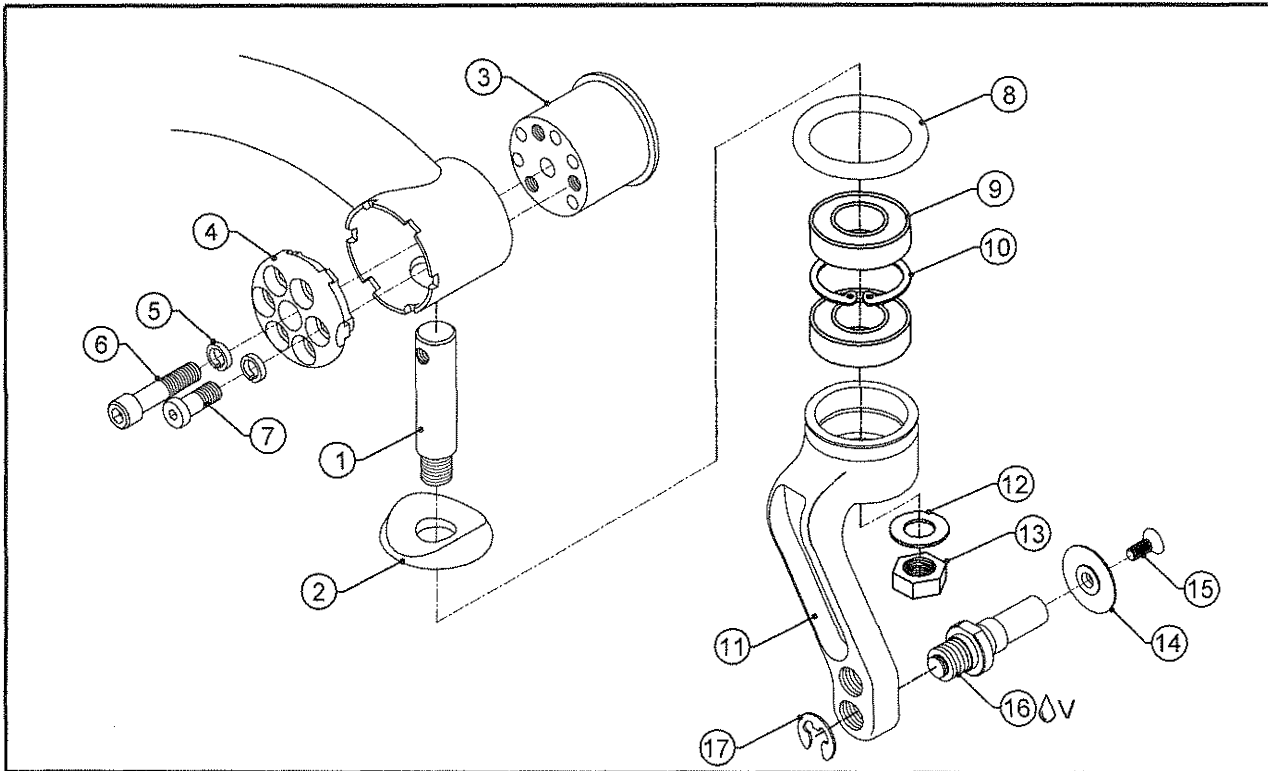

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104292	BEARING FORK STEM
1B	1	CP104302	BEARING FORK STEM W/ 1/2" EXT
1C	1	CP104303	BEARING FORK STEM W/ 1" EXT
1D	1	CP104372	TITANIUM BEARING FORK STEM
2A	1	CP104293	CASTER CAP
2B	1	CP104304	CASTER CAP 1/2" EXT
2C	1	CP104305	CASTER CAP 1" EXT
3A	1	CP104298	CASTER MOUNT, RIGHT
3B	1	CP104299	CASTER MOUNT, LEFT
4A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
4B	1	CP104295	Z2 CASTER ENDCAP, LEFT
5	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
6	1	HD100677	1/4-28 X 1-3/16" W/ 3/4" THREAD SHCS, BLACK ZINC
7	1	HD100601	1/4-28 X 5/8" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
8	1	CP104390	BUMPER GUARD
9	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
10	1	HD100330	1-1/8" INTERNAL RETAINING RING
11A	1		MONO FORK, RIGHT
11B	1		MONO FORK, LEFT
11C	1		MONO FORK FOR 1.5" WIDE CASTER, RIGHT
11D	1		MONO FORK FOR 1.5" WIDE CASTER, LEFT
12	1	HD100353	3/8" AN WASHER STAINLESS STEEL
13	1	HD100352	3/8-24 NYLOCK NUT
14A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
14B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
15A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW, WITH THREAD PATCH
15B	1	HD100339	10-32 X 5/8" FLAT HEAD CAP SCREW, BLACK ZINC, WITH THREAD PATCH
16 V	1	CP104308	SINGLE SIDED FORK AXLE
17	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
8-10, 11A, 16, 17		AS110937	MONO FORK SUB ASSY, RIGHT
8-10, 11B, 16, 17		AS110938	MONO FORK SUB ASSY, LEFT
8-10, 11C, 16, 17		AS110939	MONO FORK SUB ASSY FOR 1.5" WIDE CASTER, RIGHT
8-10, 11D, 16, 17		AS110940	MONO FORK SUB ASSY FOR 1.5" WIDE CASTER, LEFT

ASSEMBLIES ON NEXT PAGE



MONO FORK ASSEMBLY (CONT.)


TiLite™ ZRA (SERIES 2), TRA (SERIES 2)

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A,2A,3A,4A-10,11A-14A,15A,16,17		AS110677	ZRA MONO FORK FOR SILVER CASTER, RIGHT
1A,2A,3A,4A-10,11A-14B,15B,16,17		AS110962	ZRA MONO FORK FOR BLACK CASTER, RIGHT
1A,2A,3B,4B-10,11B-14A,15A,16,17		AS110678	ZRA MONO FORK FOR SILVER CASTER, LEFT
1A,2A,3B,4B-10,11B-14B,15B,16,17		AS110963	ZRA MONO FORK FOR BLACK CASTER, LEFT
1B,2B,3A,4A-10,11A-14A,15A,16,17		AS110680	ZRA MONO FORK 1/2" EXT FOR SILVER CASTER, RIGHT
1B,2B,3A,4A-10,11A-14B,15B,16,17		AS110964	ZRA MONO FORK 1/2" EXT FOR BLACK CASTER, RIGHT
1B,2B,3B,4B-10,11B-14A,15A,16,17		AS110681	ZRA MONO FORK 1/2" EXT FOR SILVER CASTER, LEFT
1B,2B,3B,4B-10,11B-14B,15B,16,17		AS110965	ZRA MONO FORK 1/2" EXT FOR BLACK CASTER, LEFT
1C,2C,3A,4A-10,11A-14A,15A,16,17		AS110683	ZRA MONO FORK 1" EXT FOR SILVER CASTER, RIGHT
1C,2C,3A,4A-10,11A-14B,15B,16,17		AS110966	ZRA MONO FORK 1" EXT FOR BLACK CASTER, RIGHT
1C,2C,3B,4B-10,11B-14A,15A,16,17		AS110684	ZRA MONO FORK 1" EXT FOR SILVER CASTER, LEFT
1C,2C,3B,4B-10,11B-14B,15B,16,17		AS110967	ZRA MONO FORK 1" EXT FOR BLACK CASTER, LEFT
1D,2A,3A,4A-10,11A-14A,15A,16,17		AS110913	ZRA MONO FORK, TI STEM FOR SILVER CASTER, RIGHT
1D,2A,3A,4A-10,11A-14B,15B,16,17		AS110968	ZRA MONO FORK, TI STEM FOR BLACK CASTER, RIGHT
1D,2A,3B,4B-10,11B-14A,15A,16,17		AS110914	ZRA MONO FORK, TI STEM FOR SILVER CASTER, LEFT
1D,2A,3B,4B-10,11B-14B,15B,16,17		AS110969	ZRA MONO FORK, TI STEM FOR BLACK CASTER, LEFT
1D,2A,3A,4A-10,11C-14A,15A,16,17		AS115294	ZRA MONO FORK, TI STEM FOR 1.5" WIDE SILVER CASTER, RIGHT
1D,2A,3B,4B-10,11D-14A,15A,16,17		AS115295	ZRA MONO FORK, TI STEM FOR 1.5" WIDE SILVER CASTER, LEFT
1D,2A,3A,4A-10,11C-14B,15B,16,17		AS117434	ZRA MONO FORK, TI STEM FOR 1.5" WIDE BLACK CASTER, RIGHT
1D,2A,3B,4B-10,11D-14B,15B,16,17		AS117435	ZRA MONO FORK, TI STEM FOR 1.5" WIDE BLACK CASTER, LEFT
1A,2A,3A,4A-10,11C-14B,15B,16,17		AS110887	ZRA MONO FORK FOR 1.5" WIDE BLACK CASTER, RIGHT
1A,2A,3B,4B-10,11D-14B,15B,16,17		AS110888	ZRA MONO FORK FOR 1.5" WIDE BLACK CASTER, LEFT
1B,2B,3A,4A-10,11C-14B,15B,16,17		AS110889	ZRA MONO FORK FOR 1.5" WIDE BLACK CASTER 1/2" EXT, RIGHT
1B,2B,3B,4B-10,11D-14B,15B,16,17		AS110890	ZRA MONO FORK FOR 1.5" WIDE BLACK CASTER 1/2" EXT, LEFT
1C,2C,3A,4A-10,11C-14B,15B,16,17		AS110891	ZRA MONO FORK FOR 1.5" WIDE BLACK CASTER 1" EXT, RIGHT
1C,2C,3B,4B-10,11D-14B,15B,16,17		AS110892	ZRA MONO FORK FOR 1.5" WIDE BLACK CASTER 1" EXT, LEFT



BEARING FORK ASSEMBLY

TiLite™ ZRA (SERIES 2), TRA (SERIES 2)

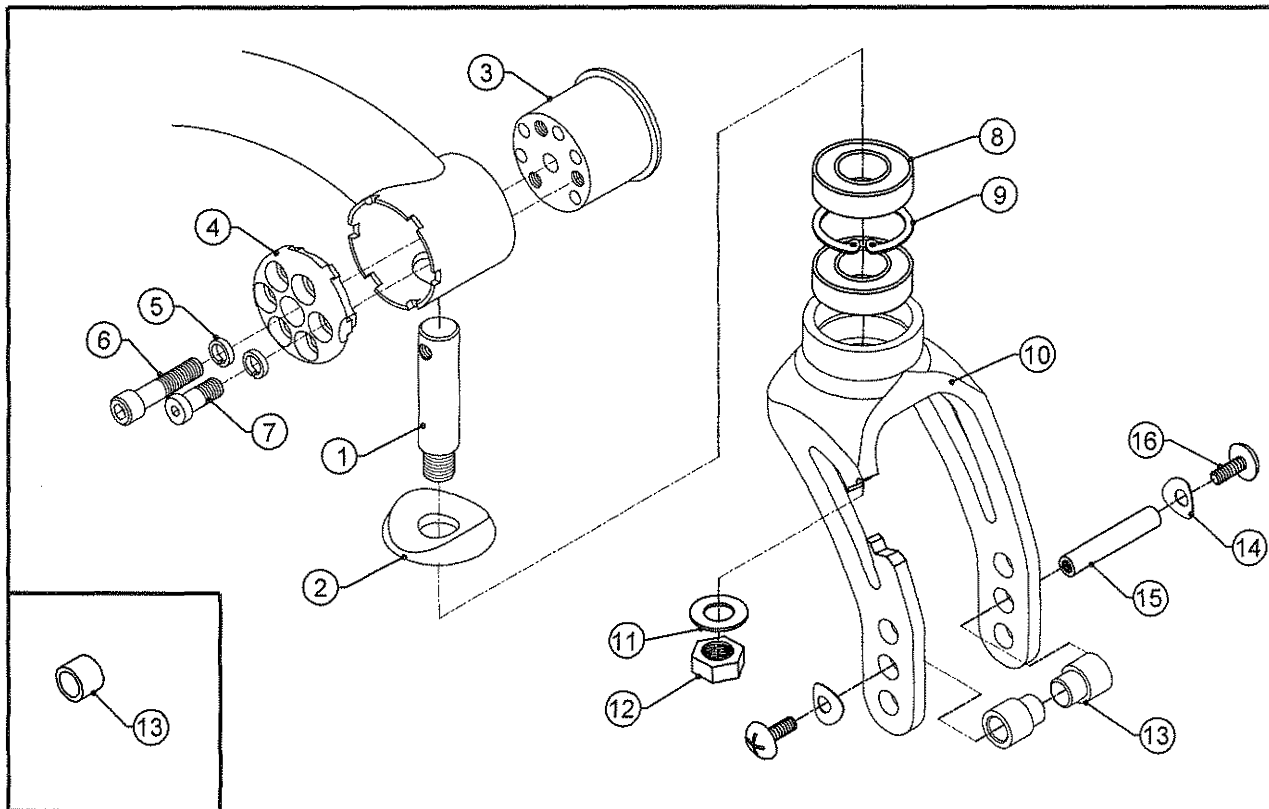

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104292	BEARING FORK STEM
1B	1	CP104302	BEARING FORK STEM W/ 1/2" EXT
1C	1	CP104303	BEARING FORK STEM W/ 1" EXT
1D	1	CP104372	TITANIUM BEARING FORK STEM
2A	1	CP104293	CASTER CAP
2B	1	CP104304	CASTER CAP 1/2" EXT
2C	1	CP104305	CASTER CAP 1" EXT
3A	1	CP104298	CASTER MOUNT, RIGHT
3B	1	CP104299	CASTER MOUNT, LEFT
4A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
4B	1	CP104295	Z2 CASTER ENDCAP, LEFT
5	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
6	1	HD100677	1/4-28 X 1-3/16" W/ 3/4" THREAD SHCS, BLACK ZINC
7	1	HD100601	1/4-28 X 5/8" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
8	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
9	1	HD100330	1-1/8" INTERNAL RETAINING RING
10	1	CP106957	BEARING FORK FOR 1.5" WIDE CASTER
11	1	HD100353	3/8" AN WASHER STAINLESS STEEL
12	1	HD100352	3/8-24 NYLOCK NUT
13	2	TABLE ONE	CASTER SPACER
14	2	CP104375	C SPRING WASHER
15	1	CP103041	CASTER AXLE LONG
16	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE W/ THREAD PATCH
14-16		AS116086	HARDWARE KIT FOR BEARING FORK ASSY.
1A,2A,3A,4A-16		TABLE ONE	STD BEARING FORK ASSEMBLY, RIGHT
1A,2A,3B,4B-16		TABLE ONE	STD BEARING FORK ASSEMBLY, LEFT
1B,2B,3A,4A-16		TABLE ONE	STD BEARING FORK ASSEMBLY 1/2" EXT, RIGHT
1B,2B,3B,4B-16		TABLE ONE	STD BEARING FORK ASSEMBLY 1/2" EXT, LEFT
1C,2C,3A,4A-16		TABLE ONE	STD BEARING FORK ASSEMBLY 1" EXT, RIGHT
1C,2C,3B,4B-16		TABLE ONE	STD BEARING FORK ASSEMBLY 1" EXT, LEFT
1D,2A,3A,4A-16		TABLE TWO	STD BEARING FORK ASSEMBLY, TI STEM, RIGHT
1D,2A,3B,4B-16		TABLE TWO	STD BEARING FORK ASSEMBLY, TI STEM, LEFT

TABLES ON NEXT PAGE



Parts Manual

BEARING FORK ASSEMBLY (CONT.)

TiLite™ ZRA (SERIES 2), TRA (SERIES 2)

TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK RIGHT	STD FORK LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1" EXT RIGHT	STD FORK 1" EXT LEFT
3" x .75" MICRO	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691
3" x .75" VOLCANIC	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691
4" x .75" PLASTIC W/ POLY	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691
4" x .75" VOLCANIC	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691
4" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
5" x 1" PLASTIC W/ POLY	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691
5" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
6" x 1" PLASTIC W/ POLY	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691
6" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS110692	AS110693	AS110694	AS110695	AS110696	AS110697
6" x 1.25" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS110686	AS110687	AS110688	AS110689	AS110690	AS110691


TABLE TWO - TITANIUM STEM FORK ASSEMBLIES

	STD FORK RIGHT	STD FORK LEFT
3" x .75" MICRO	AS110915	AS110916
3" x .75" VOLCANIC	AS110915	AS110916
4" x .75" PLASTIC W/ POLY	AS110915	AS110916
4" x .75" VOLCANIC	AS110915	AS110916
4" x .75" LITESPEED W/ POLY	AS110917	AS110918
4" x 1.5" LITESPEED W/ SOFT ROLL	AS110917	AS110918
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	AS110915	AS110916
5" x 1" PLASTIC W/ POLY	AS110915	AS110916
5" x .75" LITESPEED W/ POLY	AS110917	AS110918
5" x 1.5" LITESPEED W/ SOFT ROLL	AS110917	AS110918
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	AS110917	AS110918
6" x 1" PLASTIC W/ POLY	AS110915	AS110916
6" x 1.25" PLASTIC W/ PNEUMATIC	AS110915	AS110916
6" x .75" LITESPEED W/ POLY	AS110917	AS110918
6" x 1.5" LITESPEED W/ SOFT ROLL	AS110917	AS110918
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	AS110917	AS110918
6" x 1.25" LITESPEED W/ PNEUMATIC	AS110915	AS110916

*ASSEMBLIES DO NOT INCLUDE CASTER WHEELS

MONO FORK ASSEMBLY

TiLite™ ZR (SERIES 2), TR (SERIES 3)

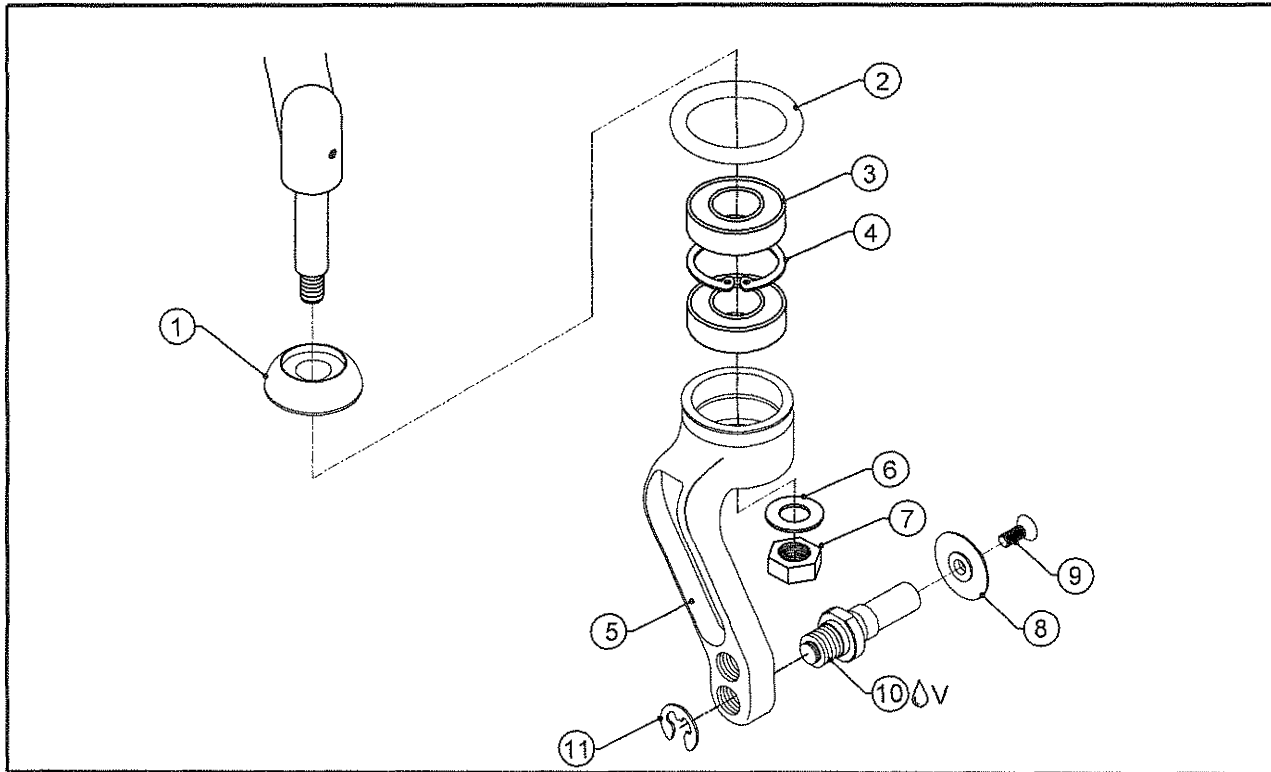


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103744	CASTER CAP, MONO FORK
2	1	CP104390	BUMPER GUARD
3	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
4	1	HD100330	1-1/8" INTERNAL RETAINING RING
5A	1		MONO FORK, RIGHT
5B	1		MONO FORK, LEFT
5C	1		MONO FORK FOR 1.5" WIDE CASTER, RIGHT
5D	1		MONO FORK FOR 1.5" WIDE CASTER, LEFT
6	1	HD100353	3/8" AN WASHER STAINLESS STEEL
7	1	HD100352	3/8-24 NYLOCK NUT
8A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
8B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
9A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW, WITH THREAD PATCH
9B	1	HD100339	10-32 X 5/8" FLAT HEAD CAP SCREW, BLACK ZINC, WITH THREAD PATCH
10A	1	CP104308	SINGLE SIDED FORK AXLE
10B	1	CP104717	SINGLE SIDED FORK AXLE (FROG LEG)
11	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
2-4, 5A, 10A, 11		AS110937	MONO FORK SUB ASSY, RIGHT
2-4, 5B, 10A, 11		AS110938	MONO FORK SUB ASSY, LEFT
2-4, 5C, 10B, 11		AS110939	MONO FORK SUB ASSY FOR 1.5" WIDE CASTER, RIGHT
2-4, 5D, 10B, 11		AS110940	MONO FORK SUB ASSY FOR 1.5" WIDE CASTER, LEFT
1-4, 5A, 6-8A, 9A, 10A, 11		AS110798	MONO FORK ASSEMBLY FOR SILVER CASTER, RIGHT
1-4, 5A, 6-8B, 9B, 10A, 11		AS110960	MONO FORK ASSEMBLY FOR BLACK CASTER, RIGHT
1-4, 5B, 6-8A, 9A, 10A, 11		AS110799	MONO FORK ASSEMBLY FOR SILVER CASTER, LEFT
1-4, 5B, 6-8B, 9B, 10A, 11		AS110961	MONO FORK ASSEMBLY FOR BLACK CASTER, LEFT
1-4, 5C, 6-8A, 9A, 10B, 11		AS117442	MONO FORK ASSEMBLY FOR 1.5" WIDE SILVER CASTER, RIGHT
1-4, 5D, 6-8A, 9A, 10B, 11		AS117443	MONO FORK ASSEMBLY FOR 1.5" WIDE SILVER CASTER, LEFT
1-4, 5C, 6-8B, 9B, 10B, 11		AS110893	MONO FORK ASSEMBLY FOR 1.5" WIDE BLACK CASTER, RIGHT
1-4, 5D, 6-8B, 9B, 10B, 11		AS110894	MONO FORK ASSEMBLY FOR 1.5" WIDE BLACK CASTER, LEFT



Parts Manual

BEARING FORK ASSEMBLY

TiLite™ ZR (SERIES 2), TR (SERIES 3)

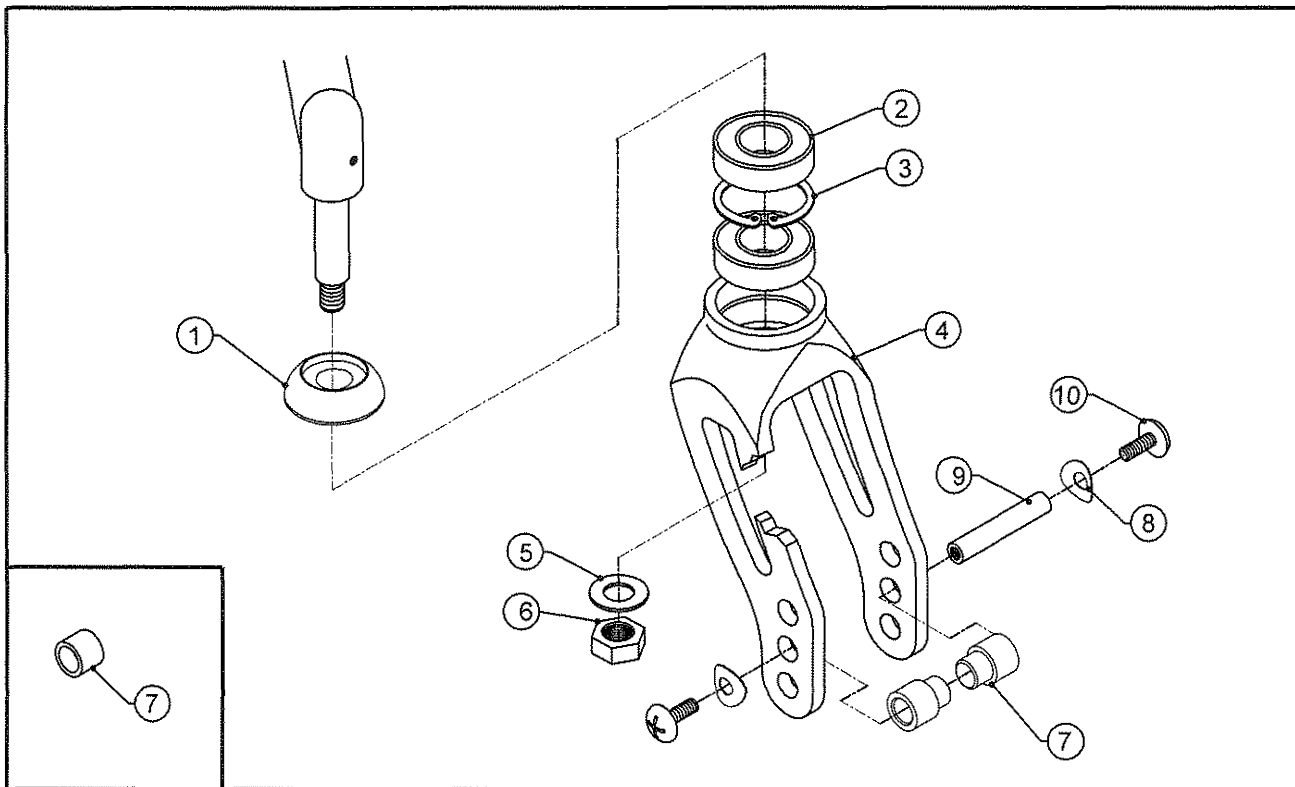


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103744	CASTER CAP, MONO FORK
2	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
3	1	HD100330	1-1/8" INTERNAL RETAINING RING
4A	1	CP103566	BEARING FORK FOR 1.5" WIDE CASTER
4B	1	CP103567	BEARING FORK FOR 1.5" WIDE CASTER SHORT
5	1	HD100353	3/8" AN WASHER STAINLESS STEEL
6	1	HD100352	3/8-24 NYLOCK NUT
7	2	TABLE ONE	CASTER SPACER
8	2	CP104375	C SPRING WASHER
9	1	CP103041	CASTER AXLE LONG
10	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
8,9,10		AS116086	HARDWARE KIT FOR BEARING FORK ASSY
1-3,4A,5-10		TABLE ONE	STD BEARING FORK ASSEMBLY
1-3,4B,5-10		TABLE ONE	SHORT BEARING FORK ASSEMBLY
		TABLE ONE	- SPACERS AND FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK	SHORT FORK
3" x .75" MICRO	CP103945	CP103945	AS110800	AS110801
3" x .75" VOLCANIC	CP103945	CP103945	AS110800	AS110801
4" x .75" PLASTIC W/ POLY	CP103945	CP103945	AS110800	AS110801
4" x .75" VOLCANIC	CP103945	CP103945	AS110800	AS110801
4" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS110802	AS110803
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS110802	AS110803
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS110802	AS110803
5" x 1" PLASTIC W/ POLY	CP103945	CP103945	AS110800	N/A
5" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS110802	N/A
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS110802	N/A
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS110802	N/A
6" x 1" PLASTIC W/ POLY	CP103945	CP103945	AS110800	N/A
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS110800	N/A
6" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS110802	N/A
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS110802	N/A
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS110802	N/A
6" x 1.25" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS110800	N/A

WIDE BEARING FORK ASSEMBLY

TiLite™ AERO Z

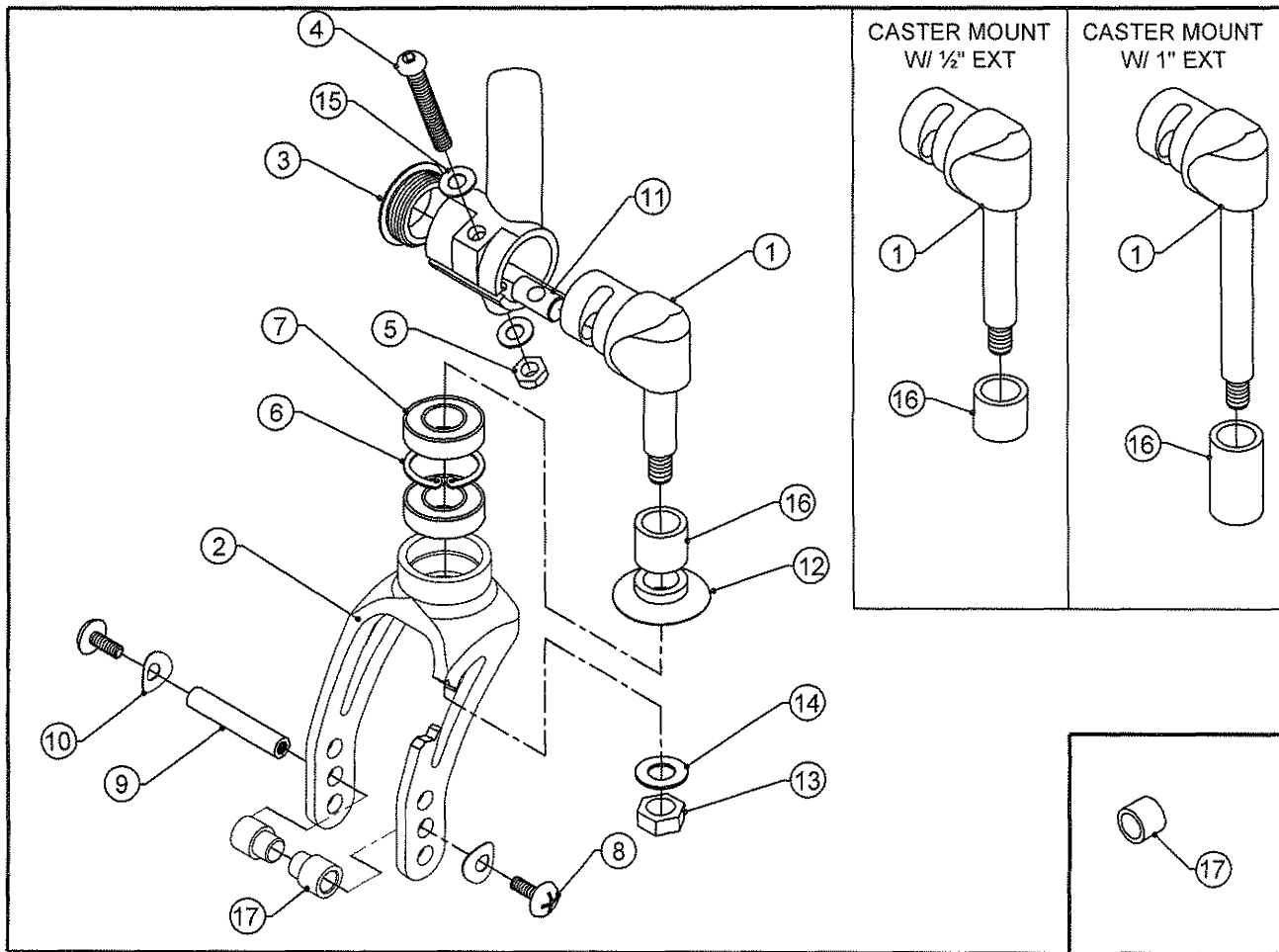

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS102842	CASTER MOUNT FOR BEARING FORK RIGHT ASSEMBLY
1B	1	AS102843	CASTER MOUNT FOR BEARING FORK LEFT ASSEMBLY
1C	1	AS102844	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT RIGHT ASSEMBLY
1D	1	AS102845	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT LEFT ASSEMBLY
1E	1	AS102846	CASTER MOUNT FOR BEARING FORK W/ 1" EXT RIGHT ASSEMBLY
1F	1	AS102847	CASTER MOUNT FOR BEARING FORK W/ 1" EXT LEFT ASSEMBLY
2A	1	CP106957	BEARING FORK FOR 1.5" WIDE CASTER
2B	1	CP106956	BEARING FORK FOR 1.5" WIDE CASTER SHORT
3	1	CP102064	ENDCAP 1.25 OD X .125 WALL
4	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
5	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
6	1	HD100330	1-1/8" INTERNAL RETAINING RING
7	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
8	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
9	1	CP103041	CASTER AXLE LONG
10	2	CP104375	C SPRING WASHER
11	1	CP102142	CASTER MOUNT DOWEL Ø.437
12	1	CP102048	CASTER CAP BEARING FORK
13	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
14	1	HD100353	3/8" AN WASHER STAINLESS STEEL
15	2	HD100402	WASHER 1/4" LIGHT AN (.265 ID X .500 OD X .032 THK)
16A	1	CP100346	CASTER EXTENSION 1/2"
16B	1	CP100347	CASTER EXTENSION 1"
8,9,10		AS116086	HARDWARE KIT FOR BEARING FORK ASSY.
17	2	TABLE ONE	CASTER SPACER

ASSEMBLIES ON NEXT PAGE

WIDE BEARING FORK ASSEMBLY (CONT.)

TiLite™ AERO Z

ITEM	QTY	PART NO.	DESCRIPTION
1B,2A-14,16	TABLE ONE	AERO Z WIDE BEARING FORK ASSEMBLY LEFT	
1A,2A-14,16	TABLE ONE	AERO Z WIDE BEARING FORK ASSEMBLY RIGHT	
1D,2A-14,15A,16	TABLE ONE	AERO Z WIDE BEARING FORK ASSEMBLY LEFT W/ 1/2" EXT	
1C,2A-14,15A,16	TABLE ONE	AERO Z WIDE BEARING FORK ASSEMBLY RIGHT W/ 1/2" EXT	
1F,2A-14,15B,16	TABLE ONE	AERO Z WIDE BEARING FORK ASSEMBLY LEFT W/ 1" EXT	
1E,2A-14,15B,16	TABLE ONE	AERO Z WIDE BEARING FORK ASSEMBLY RIGHT W/ 1" EXT	
1D,2B-14,16	TABLE TWO	AERO Z WIDE SHORT BEARING FORK ASSEMBLY LEFT	
1C,2B-14,16	TABLE TWO	AERO Z WIDE SHORT BEARING FORK ASSEMBLY RIGHT	
1F,2B-14,15A,16	TABLE TWO	AERO Z WIDE SHORT BEARING FORK ASSEMBLY LEFT W/ 1/2" EXT	
1E,2B-14,15A,16	TABLE TWO	AERO Z WIDE SHORT BEARING FORK ASSEMBLY RIGHT W/ 1/2" EXT	
1F,2B-14,15B,16	TABLE TWO	AERO Z WIDE SHORT BEARING FORK ASSEMBLY LEFT W/ 1" EXT	
1E,2B-14,15B,16	TABLE TWO	AERO Z WIDE SHORT BEARING FORK ASSEMBLY RIGHT W/ 1" EXT	

TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK LEFT	STD FORK RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1" EXT LEFT	STD FORK 1" EXT RIGHT
3" MICRO	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280
3" VOLCANIC	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280
4" PLASTIC W/ POLY	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280
4" VOLCANIC	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280
4" LITESPEED W/ POLY	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
5" PLASTIC W/ POLY	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280
5" LITESPEED W/ POLY	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
6" PLASTIC W/ POLY	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280
6" LITESPEED W/ POLY	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111366	AS111367	AS111368	AS111369	AS111370	AS111371
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS104271	AS104272	AS104275	AS104276	AS104279	AS104280


TABLE TWO - SPACERS AND SHORT FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	SHORT FORK LEFT	SHORT FORK RIGHT	SHORT FORK 1/2" EXT LEFT	SHORT FORK 1/2" EXT RIGHT	SHORT FORK 1" EXT LEFT	SHORT FORK 1" EXT RIGHT
3" MICRO	CP103945	CP103945	AS104298	AS104299	AS104300	AS104301	AS104302	AS104303
3" VOLCANIC	CP103945	CP103945	AS104298	AS104299	AS104300	AS104301	AS104302	AS104303
4" PLASTIC W/ POLY	CP103945	CP103945	AS104298	AS104299	AS104300	AS104301	AS104302	AS104303
4" VOLCANIC	CP103945	CP103945	AS104298	AS104299	AS104300	AS104301	AS104302	AS104303
4" LITESPEED W/ POLY	CP104405	CP104406	AS104298	AS104299	AS104300	AS104301	AS104302	AS104303
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111372	AS111373	AS111374	AS111375	AS111376	AS111377
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111372	AS111373	AS111374	AS111375	AS111376	AS111377



SLIPSTREAM FORK ASSEMBLY

TiLite™ AERO Z (SERIES 2), TWIST, AERO T

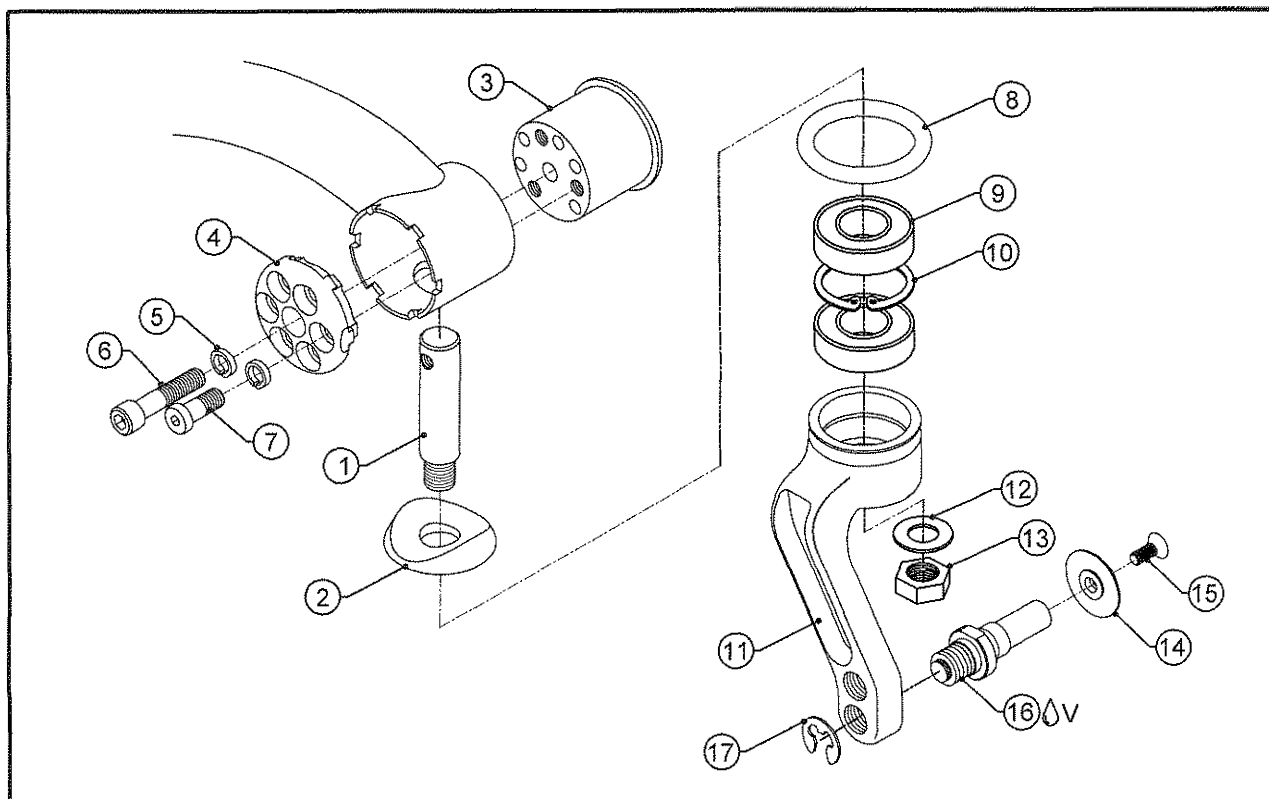

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104292	BEARING FORK STEM
1B	1	CP104302	BEARING FORK STEM W/ 1/2" EXT
1C	1	CP104303	BEARING FORK STEM W/ 1" EXT
1D	1	CP104372	TITANIUM BEARING FORK STEM
2A	1	CP105482	CASTER CAP
2B	1	CP105483	CASTER CAP 1/2" EXT
2C	1	CP105484	CASTER CAP 1" EXT
3A	1	CP104298	CASTER MOUNT, RIGHT
3B	1	CP104299	CASTER MOUNT, LEFT
4A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
4B	1	CP104295	Z2 CASTER ENDCAP, LEFT
5	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
6	1	HD100676	1/4-28 X 1-7/16" W/ 3/4" THREAD SHCS, BLACK ZINC
7	1	HD100661	1/4-28 X 29/32 W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
8	1	CP104390	BUMPER GUARD
9	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
10	1	HD100330	1-1/8" INTERNAL RETAINING RING
11A	1		MONO FORK, RIGHT
11B	1		MONO FORK, LEFT
11C	1		MONO FORK FOR 1.5" WIDE CASTER, RIGHT
11D	1		MONO FORK FOR 1.5" WIDE CASTER, LEFT
12	1	HD100353	3/8" AN WASHER STAINLESS STEEL
13	1	HD100352	3/8-24 NYLOCK NUT
14A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
14B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
15A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW, WITH PATCH
15B	1	HD100339	10-32 X 5/8" FLAT HEAD CAP SCREW, BLACK ZINC, WITH PATCH
16A V	1	CP104308	SINGLE SIDED FORK AXLE
16B V	1	CP104717	SINGLE SIDED FORK AXLE (1.5" WIDE CASTER)
17	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
8-10,11A,16A,17		AS110937	MONO FORK SUB ASSY, RIGHT
8-10,11B,16A,17		AS110938	MONO FORK SUB ASSY, LEFT
8-10,11C,16B,17		AS110939	MONO FORK SUB ASSY FOR 1.5" WIDE CASTER, RIGHT
8-10,11D,16B,17		AS110940	MONO FORK SUB ASSY FOR 1.5" WIDE CASTER, LEFT

ASSEMBLIES ON NEXT PAGE


SLIPSTREAM FORK ASSEMBLY (CONT.)

TiLite™ AERO Z (SERIES 2), TWIST, AERO T

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A,2A,3A,4A-10,11A-14A,15A,16A,17		AS111454	AERO Z MONO FORK FOR SILVER CASTER, RIGHT
1A,2A,3A,4A-10,11A-14B,15B,16A,17		AS111455	AERO Z MONO FORK FOR BLACK CASTER, RIGHT
1A,2A,3B,4B-10,11B-14A,15A,16A,17		AS111456	AERO Z MONO FORK FOR SILVER CASTER, LEFT
1A,2A,3B,4B-10,11B-14B,15B,16A,17		AS111457	AERO Z MONO FORK FOR BLACK CASTER, LEFT
1B,2B,3A,4A-10,11A-14A,15A,16A,17		AS111458	AERO Z MONO FORK 1/2" EXT FOR SILVER CASTER, RIGHT
1B,2B,3A,4A-10,11A-14B,15B,16A,17		AS111459	AERO Z MONO FORK 1/2" EXT FOR BLACK CASTER, RIGHT
1B,2B,3B,4B-10,11B-14A,15A,16A,17		AS111460	AERO Z MONO FORK 1/2" EXT FOR SILVER CASTER, LEFT
1B,2B,3B,4B-10,11B-14B,15B,16A,17		AS111461	AERO Z MONO FORK 1/2" EXT FOR BLACK CASTER, LEFT
1C,2C,3A,4A-10,11A-14A,15A,16A,17		AS111462	AERO Z MONO FORK 1" EXT FOR SILVER CASTER, RIGHT
1C,2C,3A,4A-10,11A-14B,15B,16A,17		AS111463	AERO Z MONO FORK 1" EXT FOR BLACK CASTER, RIGHT
1C,2C,3B,4B-10,11B-14A,15A,16A,17		AS111464	AERO Z MONO FORK 1" EXT FOR SILVER CASTER, LEFT
1C,2C,3B,4B-10,11B-14B,15B,16A,17		AS111465	AERO Z MONO FORK 1" EXT FOR BLACK CASTER, LEFT
1D,2A,3A,4A-10,11A-14A,15A,16A,17		AS111466	AERO Z MONO FORK, TI STEM FOR SILVER CASTER, RIGHT
1D,2A,3A,4A-10,11A-14B,15B,16A,17		AS111467	AERO Z MONO FORK, TI STEM FOR BLACK CASTER, RIGHT
1D,2A,3B,4B-10,11B-14A,15A,16A,17		AS111468	AERO Z MONO FORK, TI STEM FOR SILVER CASTER, LEFT
1D,2A,3B,4B-10,11B-14B,15B,16A,17		AS111469	AERO Z MONO FORK, TI STEM FOR BLACK CASTER, LEFT
1D,2A,3A,4A-10,11C-14A,15A,16B,17		AS115292	AERO Z MONO FORK, TI STEM FOR 1.5" WIDE SILVER CASTER, RIGHT
1D,2A,3B,4B-10,11D-14A,15A,16B,17		AS115293	AERO Z MONO FORK, TI STEM FOR 1.5" WIDE SILVER CASTER, LEFT
1D,2A,3A,4A-10,11C-14B,15B,16B,17		AS117444	AERO Z MONO FORK, TI STEM FOR 1.5" WIDE BLACK CASTER, RIGHT
1D,2A,3B,4B-10,11D-14B,15B,16B,17		AS117445	AERO Z MONO FORK, TI STEM FOR 1.5" WIDE BLACK CASTER, LEFT
1A,2A,3A,4A-10,11C-14B,15B,16B,17		AS111470	AERO Z MONO FORK FOR 1.5" WIDE BLACK CASTER, RIGHT
1A,2A,3B,4B-10,11D-14B,15B,16B,17		AS111471	AERO Z MONO FORK FOR 1.5" WIDE BLACK CASTER, LEFT
1B,2B,3A,4A-10,11C-14B,15B,16B,17		AS111472	AERO Z MONO FORK FOR 1.5" WIDE BLACK CASTER 1/2" EXT, RIGHT
1B,2B,3B,4B-10,11D-14B,15B,16B,17		AS111473	AERO Z MONO FORK FOR 1.5" WIDE BLACK CASTER 1/2" EXT, LEFT
1C,2C,3A,4A-10,11C-14B,15B,16B,17		AS111474	AERO Z MONO FORK FOR 1.5" WIDE BLACK CASTER 1" EXT, RIGHT
1C,2C,3B,4B-10,11D-14B,15B,16B,17		AS111475	AERO Z MONO FORK FOR 1.5" WIDE BLACK CASTER 1" EXT, LEFT



TiLite™ AERO Z (SERIES 2), TWIST, AERO T

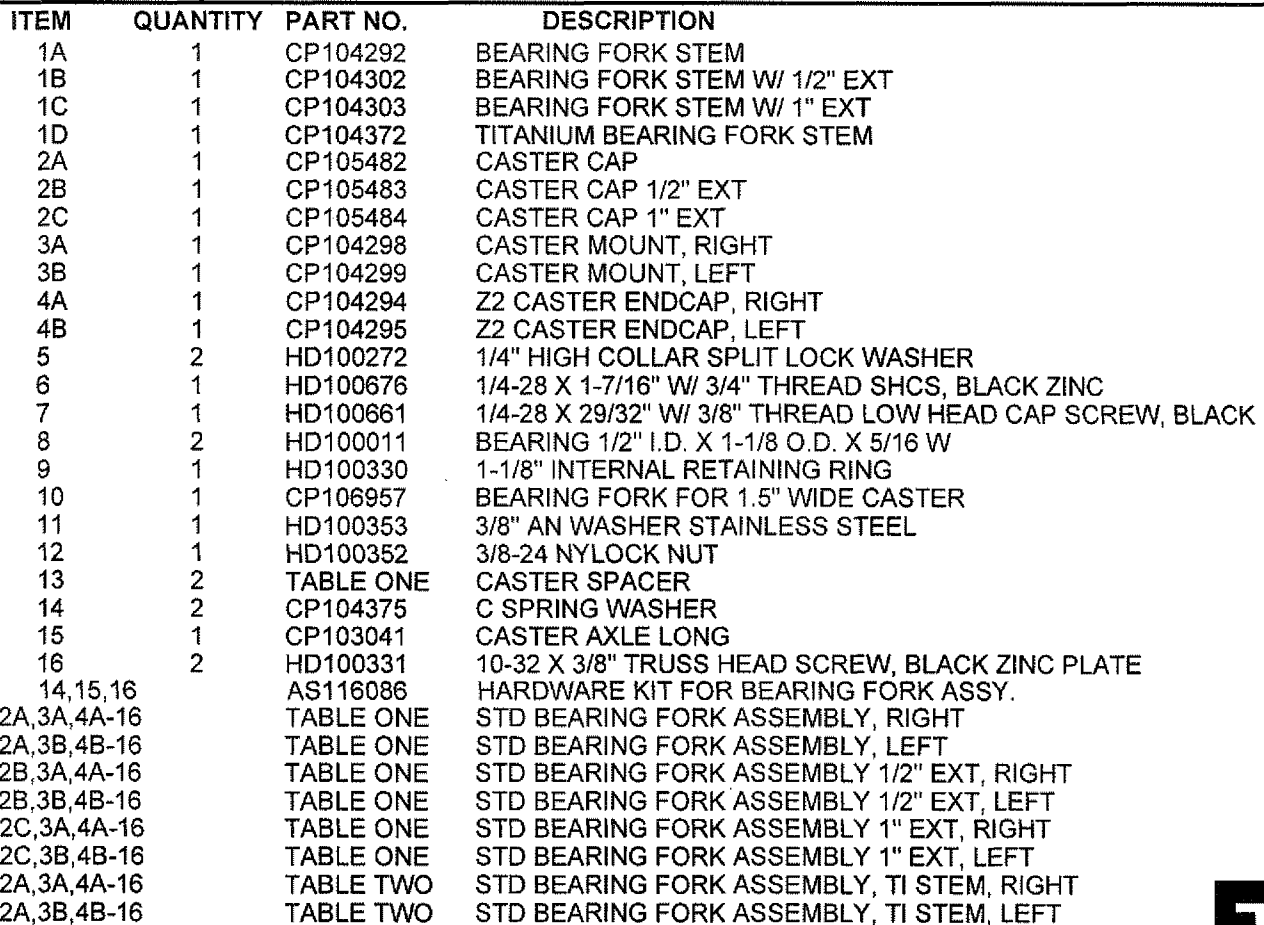

PM0712 REV 1 10/1/13 ACN

BEARING FORK ASSEMBLY (CONT.)

TiLite™ AERO Z (SERIES 2), TWIST, AERO T

TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK RIGHT	STD FORK LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1" EXT RIGHT	STD FORK 1" EXT LEFT
3" MICRO	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481
3" VOLCANIC	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481
4" PLASTIC W/ POLY	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481
4" VOLCANIC	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481
4" LITESPEED W/ POLY	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
5" PLASTIC W/ POLY	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481
5" LITESPEED W/ POLY	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
6" PLASTIC W/ POLY	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481
6" LITESPEED W/ POLY	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111482	AS111483	AS111484	AS111485	AS111486	AS111487
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS111476	AS111477	AS111478	AS111479	AS111480	AS111481

TABLE TWO - TITANIUM STEM FORK ASSEMBLIES


	STD FORK RIGHT	STD FORK LEFT
3" MICRO	AS111488	AS111489
3" VOLCANIC	AS111488	AS111489
4" PLASTIC W/ POLY	AS111488	AS111489
4" VOLCANIC	AS111488	AS111489
4" LITESPEED W/ POLY	AS111490	AS111491
4" x 1.5" LITESPEED W/ SOFT ROLL	AS111490	AS111491
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	AS111490	AS111491
5" PLASTIC W/ POLY	AS111488	AS111489
5" LITESPEED W/ POLY	AS111490	AS111491
5" x 1.5" LITESPEED W/ SOFT ROLL	AS111490	AS111491
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	AS111490	AS111491
6" PLASTIC W/ POLY	AS111488	AS111489
6" PLASTIC W/ PNEUMATIC	AS111488	AS111489
6" LITESPEED W/ POLY	AS111490	AS111491
6" x 1.5" LITESPEED W/ SOFT ROLL	AS111490	AS111491
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	AS111490	AS111491
6" LITESPEED W/ PNEUMATIC	AS111488	AS111489

*ASSEMBLIES DO NOT INCLUDE CASTER WHEELS



WIDE BEARING FORK ASSEMBLY

TiLite™ Aero X, 2GX, Aero R

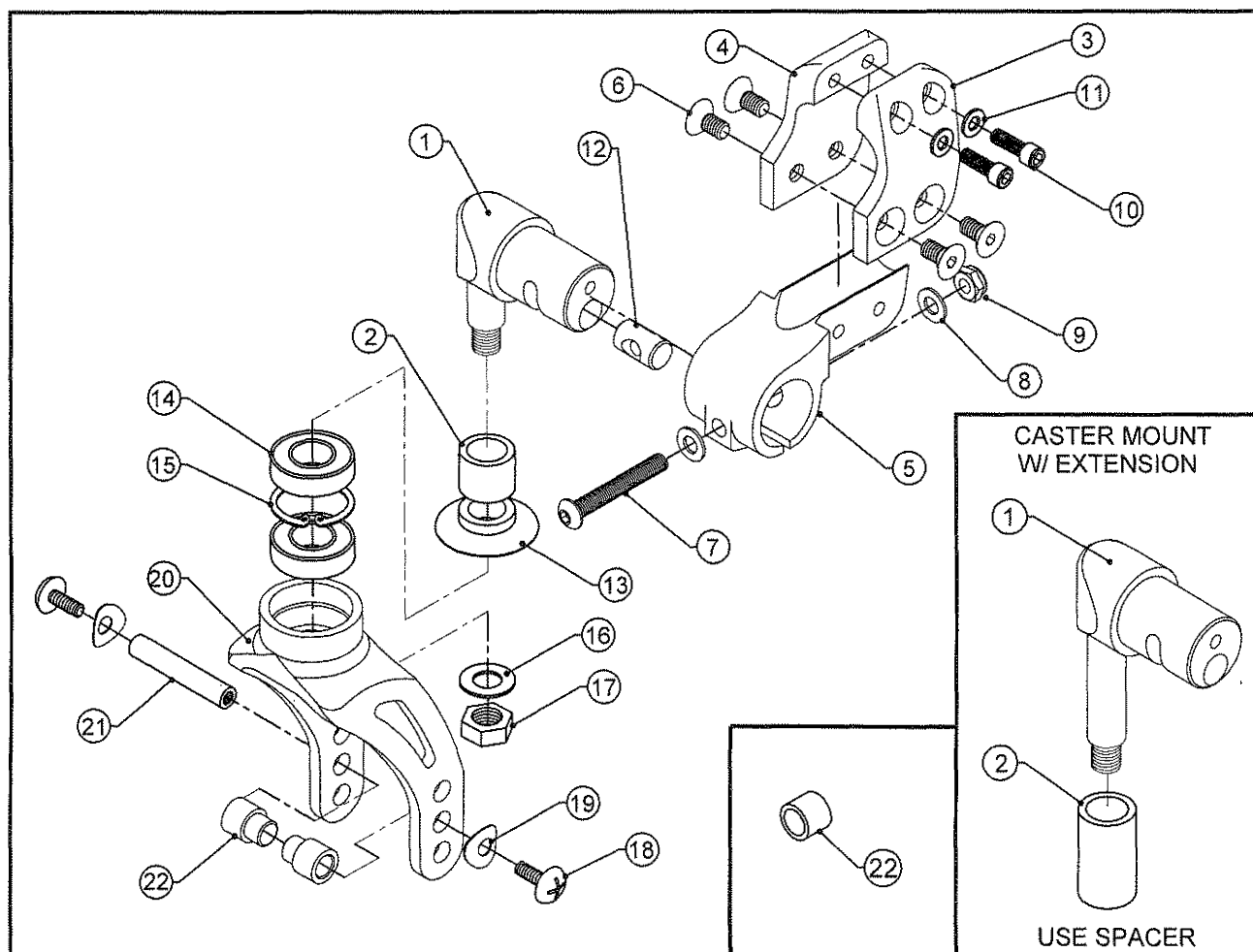

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS103237	BEARING FORK CASTER MOUNT ASSEMBLY
1B	1	AS103238	BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
1C	1	AS103239	BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
2A	1	CP100346	CASTER EXTENSION 1/2"
2B	1	CP100347	CASTER EXTENSION 1"
3A	1	CP103895	CASTER MOUNT CLAMP, COUNTERBORED, RIGHT
3B	1	CP103896	CASTER MOUNT CLAMP, COUNTERBORED, LEFT
4A	1	CP103897	CASTER MOUNT CLAMP, TAPPED, RIGHT
4B	1	CP103898	CASTER MOUNT CLAMP, TAPPED, LEFT
5	1	CP103185	CASTER MOUNT
6	4	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
7	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
8	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THK, BLACK ZINC
9	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
10	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
11	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	1	CP102142	CASTER MOUNT DOWEL Ø.437
13	1	CP102048	CASTER CAP BEARING FORK
14	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
15	1	HD100330	1-1/8" INTERNAL RETAINING RING
16	1	HD100353	3/8" AN WASHER STAINLESS STEEL
17	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
18	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
19	2	CP104375	C SPRING WASHER
20A	1	CP106957	WIDE BEARING FORK
20B	1	CP106956	WIDE BEARING FORK SHORT
20C	1	CP106958	BEARING FORK TALL
21	1	CP103041	CASTER AXLE LONG
18,19,21	1	AS116086	CASTER AXLE LONG
22	2	TABLE ONE	CASTER SPACER

ASSEMBLIES ON NEXT PAGE

WIDE BEARING FORK ASSEMBLY

TiLite™ Aero X, 2GX, Aero R

ITEM	QTY	PART NO.	DESCRIPTION
1A,3A,4A-20A,21,22		TABLE ONE	WIDE BEARING FORK STD CASTER MOUNT RIGHT
1A,3B,4B-20A,21,22		TABLE ONE	WIDE BEARING FORK STD CASTER MOUNT LEFT
1B,2A,3A, 4A-20A,21,22		TABLE ONE	WIDE BEARING FORK STD CASTER MOUNT RIGHT W/ 1/2" EXT
1B,2A,3B, 4B-20A,21,22		TABLE ONE	WIDE BEARING FORK STD CASTER MOUNT LEFT W/ 1/2" EXT
1C,2B,3A,4A-20A,21,22		TABLE ONE	WIDE BEARING FORK STD CASTER MOUNT RIGHT W/ 1" EXT
1C,2B,3B,4B-20A,21,22		TABLE ONE	WIDE BEARING FORK STD CASTER MOUNT LEFT W/1" EXT
1A,3A,4A-20B,21,22		TABLE TWO	WIDE BEARING FORK SHORT CASTER MOUNT RIGHT
1A,3B,4B-20B,21,22		TABLE TWO	WIDE BEARING FORK SHORT CASTER MOUNT LEFT
1B,2A,3A,4A-20B,21,22		TABLE TWO	WIDE BEARING FORK SHORT CASTER MOUNT RIGHT W/ 1/2" EXT
1B,2A,3B,4B-20B,21,22		TABLE TWO	WIDE BEARING FORK SHORT CASTER MOUNT LEFT W/ 1/2" EXT
1C,2B,3A,4A-20B,21,22		TABLE TWO	WIDE BEARING FORK SHORT CASTER MOUNT RIGHT W/ 1" EXT
1C,2B,3B,4B-20B,21,22		TABLE TWO	WIDE BEARING FORK SHORT CASTER MOUNT LEFT W/ 1" EXT
1A,3A,4A-20C,21,22		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER
1A,3B,4B-20C,21,22		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER
1B,2A,3A,4A-20C,21,22		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER, 1/2" EXT
1B,2A,3B,4B-20C,21,22		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER, 1/2" EXT
1C,2B,3A,4A-20C,21,22		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER, 1" EXT
1C,2B,3B,4B-20C,21,22		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER, 1" EXT

TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK RIGHT	STD FORK LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1" EXT RIGHT	STD FORK 1" EXT LEFT
3" MICRO	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384
3" VOLCANIC	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384
4" PLASTIC W/ POLY	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384
4" VOLCANIC	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384
4" LITESPEED W/ POLY	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
5" PLASTIC W/ POLY	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384
5" LITESPEED W/ POLY	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
6" PLASTIC W/ POLY	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384
6" LITESPEED W/ POLY	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111378	AS111379	AS111380	AS111381	AS111382	AS111383
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS104283	AS104380	AS104285	AS104382	AS104287	AS104384

TABLE TWO - SPACERS AND SHORT FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	SHORT FORK RIGHT	SHORT FORK LEFT	SHORT FORK 1/2" EXT RIGHT	SHORT FORK 1/2" EXT LEFT	SHORT FORK 1" EXT RIGHT	SHORT FORK 1" EXT LEFT
3" MICRO	CP103945	CP103945	AS104304	AS104386	AS104305	AS104387	AS104306	AS104388
3" VOLCANIC	CP103945	CP103945	AS104304	AS104386	AS104305	AS104387	AS104306	AS104388
4" PLASTIC W/ POLY	CP103945	CP103945	AS104304	AS104386	AS104305	AS104387	AS104306	AS104388
4" VOLCANIC	CP103945	CP103945	AS104304	AS104386	AS104305	AS104387	AS104306	AS104388
4" LITESPEED W/ POLY	CP104405	CP104406	AS111384	AS111385	AS111386	AS111387	AS111388	AS111389
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111384	AS111385	AS111386	AS111387	AS111388	AS111389
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111384	AS111385	AS111386	AS111387	AS111388	AS111389


TABLE THREE - SPACERS AND TALL FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	TALL FORK RIGHT	TALL FORK LEFT	TALL FORK 1/2" EXT RIGHT	TALL FORK 1/2" EXT LEFT	TALL FORK 1" EXT RIGHT	TALL FORK 1" EXT LEFT
3" MICRO	CP103007	CP103007	AS103473	AS104389	AS103475	AS104391	AS103477	AS104393
3" VOLCANIC	CP103007	CP103007	AS103473	AS104389	AS103475	AS104391	AS103477	AS104393
4" PLASTIC W/ POLY	CP103006	CP103006	AS103474	AS104390	AS103476	AS104392	AS103478	AS104394
4" VOLCANIC	CP103007	CP103007	AS103473	AS104389	AS103475	AS104391	AS103477	AS104393
4" LITESPEED W/ POLY	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
5" PLASTIC W/ POLY	CP103006	CP103006	AS103474	AS104390	AS103476	AS104392	AS103478	AS104394
5" LITESPEED W/ POLY	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
6" PLASTIC W/ POLY	CP103006	CP103006	AS103474	AS104390	AS103476	AS104392	AS103478	AS104394
6" PLASTIC W/ PNEUMATIC	CP103006	CP103006	AS103474	AS104390	AS103476	AS104392	AS103478	AS104394
6" LITESPEED W/ POLY	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS111390	AS111391	AS111392	AS111393	AS111394	AS111395
6" LITESPEED W/ PNEUMATIC	CP103006	CP103006	AS103474	AS104390	AS103476	AS104392	AS103478	AS104394



BEARING FORK ASSEMBLY

TiLite™ Aero X2

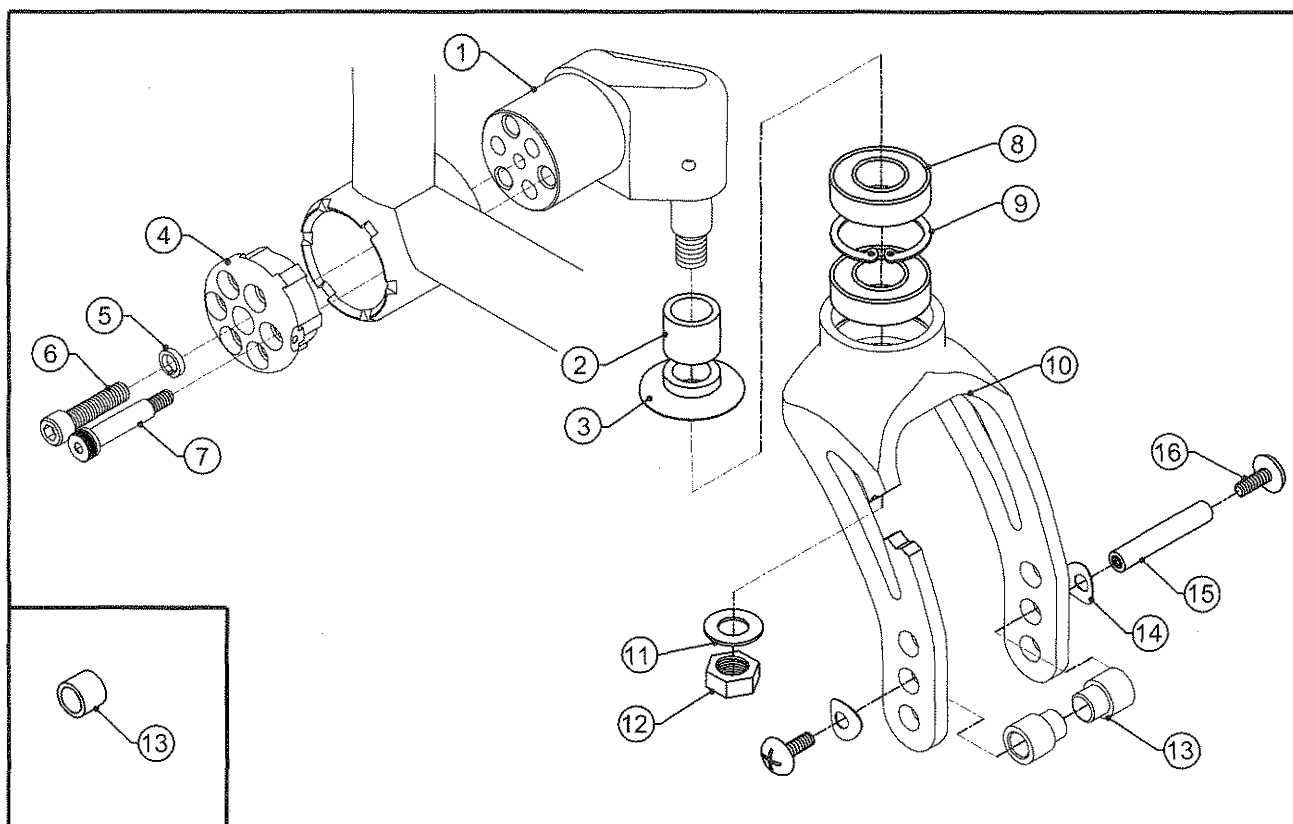

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS117267	CASTER MOUNT W/STD STEM, RIGHT
1B	1	AS117268	CASTER MOUNT W/ 1/2" EXT, RIGHT
1C	1	AS117269	CASTER MOUNT W/ 1" EXT, RIGHT
1D	1	AS117270	CASTER MOUNT W/STD STEM, LEFT
1E	1	AS117271	CASTER MOUNT W/ 1/2" EXT, LEFT
1F	1	AS117272	CASTER MOUNT W/ 1" EXT, LEFT
2A	1	CP100346	CASTER EXTENSION 1/2"
2B	1	CP100347	CASTER EXTENSION 1"
3	1	CP102048	CASTER CAP BEARING FORK
4A	1	CP107413	CASTER ENDCAP BIG RIGHT
4B	1	CP107414	CASTER ENDCAP BIG LEFT
5	1	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
6	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
7	1	HD100724	1/4" X 1" SHOULDER BOLT, THREAD PATCH
8	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
9	1	HD100330	1-1/8" INTERNAL RETAINING RING
10A	1	CP106956	BEARING FORK FOR 1.5" WIDE CASTER SHORT
10B	1	CP106957	BEARING FORK FOR 1.5" WIDE CASTER
10C	1	CP106958	BEARING FORK FOR 1.5" WIDE CASTER TALL
11	1	HD100353	3/8" AN WASHER STAINLESS STEEL
12	1	HD100352	3/8-24 NYLOCK NUT
13	2	TABLE ONE	CASTER SPACER
14	2	CP104375	C SPRING WASHER
15	1	CP103041	CASTER AXLE LONG
16	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE, THREAD PATCH

TABLES ON NEXT PAGE

BEARING FORK ASSEMBLY

TiLite™ Aero X2

ITEM	QTY	PART NO.	DESCRIPTION
1A,3,4A-10A,11-16		TABLE ONE	WIDE BEARING FORK SHORT CASTER MOUNT RIGHT
1D,3,4B-10A,11-16		TABLE ONE	WIDE BEARING FORK SHORT CASTER MOUNT LEFT
1A,3,4A-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT RIGHT
1D,3,4B-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT LEFT
1B,2A,3, 4A-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT RIGHT W/ 1/2" EXT
1E,2A,3, 4B-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT LEFT W/ 1/2" EXT
1C,2B,3,4A-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT RIGHT W/ 1" EXT
1F,2B,3,4B-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT LEFT W/ 1" EXT
1A,3,4A-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER
1D,3,4B-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER
1B,2A,3,4A-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER, 1/2" EXT
1E,2A,3,4B-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER, 1/2" EXT
1C,2B,3,4A-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER, 1" EXT
1F,2B,3,4B-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER, 1" EXT

TABLE ONE - SPACERS AND SHORT FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	SHORT FORK RIGHT	SHORT FORK LEFT
3" x .75" MICRO	CP103945	CP103945	AS117285	AS117286
3" x .75" VOLCANIC	CP103945	CP103945	AS117285	AS117286
4" x .75" PLASTIC W/ POLY	CP103945	CP103945	AS117285	AS117286
4" x .75" VOLCANIC	CP103945	CP103945	AS117285	AS117286
4" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117287	AS117288
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117287	AS117288
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117287	AS117288

TABLE TWO - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK RIGHT	STD FORK LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1" EXT RIGHT	STD FORK 1" EXT LEFT
3" x .75" MICRO	CP103945	CP103945	AS117289	AS117290	AS117291	AS117292	AS117293	AS117294
3" x .75" VOLCANIC	CP103945	CP103945	AS117289	AS117290	AS117291	AS117292	AS117293	AS117294
4" x .75" PLASTIC W/ POLY	CP103945	CP103945	AS117289	AS117290	AS117291	AS117292	AS117293	AS117294
4" x .75" VOLCANIC	CP103945	CP103945	AS117289	AS117290	AS117291	AS117292	AS117293	AS117294
4" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
5" X 1" PLASTIC W/ POLY	CP103945	CP103945	AS117289	AS117290	AS117291	AS117292	AS117293	AS117294
5" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
6" x 1" PLASTIC W/ POLY	CP103945	CP103945	AS117289	AS117290	AS117291	AS117292	AS117293	AS117294
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS117289	AS117290	AS117291	AS117292	AS117293	AS117294
6" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300
6" x 1.25" LITESPEED W/ PNEUMATIC	CP104405	CP104406	AS117295	AS117296	AS117297	AS117298	AS117299	AS117300


TABLE THREE - SPACERS AND TALL FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	TALL FORK RIGHT	TALL FORK LEFT	TALL FORK 1/2" EXT RIGHT	TALL FORK 1/2" EXT LEFT	TALL FORK 1" EXT RIGHT	TALL FORK 1" EXT LEFT
3" x .75" MICRO	CP103007	CP103007	AS117307	AS117308	AS117309	AS117310	AS117311	AS117312
3" x .75" VOLCANIC	CP103007	CP103007	AS117307	AS117308	AS117309	AS117310	AS117311	AS117312
4" x .75" PLASTIC W/ POLY	CP103006	CP103006	AS117301	AS117302	AS117303	AS117304	AS117305	AS117306
4" x .75" VOLCANIC	CP103007	CP103007	AS117307	AS117308	AS117309	AS117310	AS117311	AS117312
4" x .75" LITESPEED W/ POLY	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
5" X 1" PLASTIC W/ POLY	CP103006	CP103006	AS117301	AS117302	AS117303	AS117304	AS117305	AS117306
5" x .75" LITESPEED W/ POLY	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
6" x 1" PLASTIC W/ POLY	CP103006	CP103006	AS117301	AS117302	AS117303	AS117304	AS117305	AS117306
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103006	CP103006	AS117301	AS117302	AS117303	AS117304	AS117305	AS117306
6" x .75" LITESPEED W/ POLY	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318
6" x 1.25" LITESPEED W/ PNEUMATIC	CP104403	CP104404	AS117313	AS117314	AS117315	AS117316	AS117317	AS117318



BEARING FORK ASSEMBLY

TiLite™ 2GX (SERIES 2)

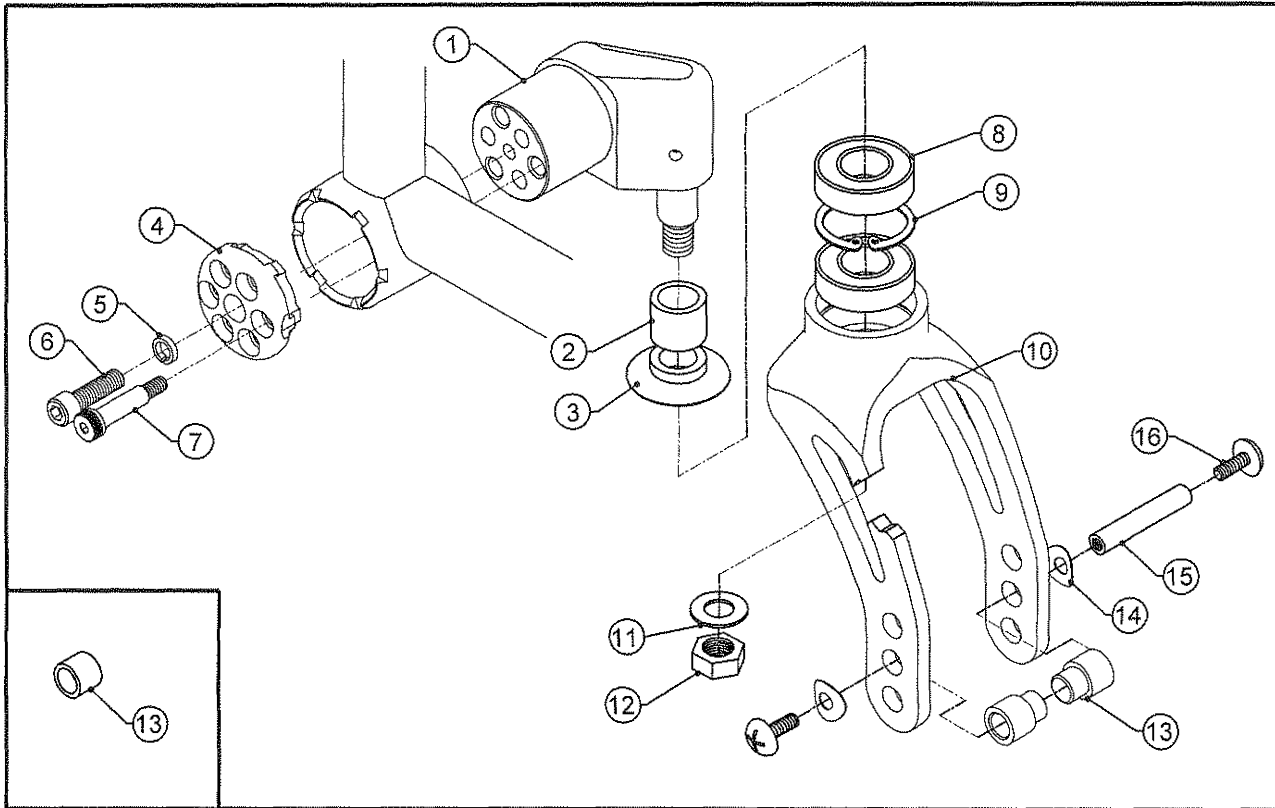

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS117267	CASTER MOUNT W/STD STEM, RIGHT
1B	1	AS117268	CASTER MOUNT W/ 1/2" EXT, RIGHT
1C	1	AS117269	CASTER MOUNT W/ 1" EXT, RIGHT
1D	1	AS117270	CASTER MOUNT W/STD STEM, LEFT
1E	1	AS117271	CASTER MOUNT W/ 1/2" EXT, LEFT
1F	1	AS117272	CASTER MOUNT W/ 1" EXT, LEFT
2A	1	CP100346	CASTER EXTENSION 1/2"
2B	1	CP100347	CASTER EXTENSION 1"
3	1	CP102048	CASTER CAP BEARING FORK
4A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
4B	1	CP104295	Z2 CASTER ENDCAP, LEFT
5	1	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
6	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
7	1	HD100733	1/4" X 3/4" SHOULDER BOLT, THREAD PATCH
8	2	HD100011	BEARING 1/2" I.D. X 1-1/8" O.D. X 5/16" W
9	1	HD100330	1-1/8" INTERNAL RETAINING RING
10A	1	CP106956	BEARING FORK FOR 1.5" WIDE CASTER SHORT
10B	1	CP106957	BEARING FORK FOR 1.5" WIDE CASTER
10C	1	CP106958	BEARING FORK FOR 1.5" WIDE CASTER TALL
11	1	HD100353	3/8" AN WASHER STAINLESS STEEL
12	1	HD100352	3/8-24 NYLOCK NUT
13	2	TABLE ONE	CASTER SPACER
14	2	CP104375	C SPRING WASHER
15	1	CP103041	CASTER AXLE LONG
16	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE, THREAD PATCH

TABLES ON NEXT PAGE



WIDE BEARING FORK ASSEMBLY

TiLite™ 2GX (SERIES 2)

ITEM	QTY	PART NO.	DESCRIPTION
1A,3,4A-10A,11-16		TABLE ONE	WIDE BEARING FORK SHORT CASTER MOUNT RIGHT
1D,3,4B-10A,11-16		TABLE ONE	WIDE BEARING FORK SHORT CASTER MOUNT LEFT
1A,3,4A-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT RIGHT
1D,3,4B-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT LEFT
1B,2A,3, 4A-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT RIGHT W/ 1/2" EXT
1E,2A,3, 4B-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT LEFT W/ 1/2" EXT
1C,2B,3,4A-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT RIGHT W/ 1" EXT
1F,2B,3,4B-10B,11-16		TABLE TWO	WIDE BEARING FORK STD CASTER MOUNT LEFT W/1" EXT
1A,3,4A-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER
1D,3,4B-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER
1B,2A,3,4A-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER, 1/2" EXT
1E,2A,3,4B-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER, 1/2" EXT
1C,2B,3,4A-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT RIGHT, 3"-4" CASTER, 1" EXT
1F,2B,3,4B-10C,11-16		TABLE THREE	WIDE BEARING FORK TALL CASTER MOUNT LEFT, 3"-4" CASTER, 1" EXT

TABLE ONE - SPACERS AND SHORT FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	SHORT FORK RIGHT	SHORT FORK LEFT
3" x .75" MICRO	CP103945	CP103945	AS117592	AS117593
3" x .75" VOLCANIC	CP103945	CP103945	AS117592	AS117593
4" x .75" PLASTIC W/ POLY	CP103945	CP103945	AS117592	AS117593
4" x .75" VOLCANIC	CP103945	CP103945	AS117592	AS117593
4" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117594	AS117595
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117594	AS117595
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117594	AS117595

TABLE TWO - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK RIGHT	STD FORK LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1" EXT RIGHT	STD FORK 1" EXT LEFT
3" x .75" MICRO	CP103945	CP103945	AS117596	AS117597	AS117598	AS117599	AS117600	AS117601
3" x .75" VOLCANIC	CP103945	CP103945	AS117596	AS117597	AS117598	AS117599	AS117600	AS117601
4" x .75" PLASTIC W/ POLY	CP103945	CP103945	AS117596	AS117597	AS117598	AS117599	AS117600	AS117601
4" x .75" VOLCANIC	CP103945	CP103945	AS117596	AS117597	AS117598	AS117599	AS117600	AS117601
4" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
5" X 1" PLASTIC W/ POLY	CP103945	CP103945	AS117596	AS117597	AS117598	AS117599	AS117600	AS117601
5" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
6" x 1" PLASTIC W/ POLY	CP103945	CP103945	AS117596	AS117597	AS117598	AS117599	AS117600	AS117601
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS117596	AS117597	AS117598	AS117599	AS117600	AS117601
6" x .75" LITESPEED W/ POLY	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607
6" x 1.25" LITESPEED W/ PNEUMATIC	CP104405	CP104406	AS117602	AS117603	AS117604	AS117605	AS117606	AS117607


TABLE THREE - SPACERS AND TALL FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	TALL FORK RIGHT	TALL FORK LEFT	TALL FORK 1/2" EXT RIGHT	TALL FORK 1/2" EXT LEFT	TALL FORK 1" EXT RIGHT	TALL FORK 1" EXT LEFT
3" x .75" MICRO	CP103007	CP103007	AS117614	AS117615	AS117616	AS117617	AS117618	AS117619
3" x .75" VOLCANIC	CP103007	CP103007	AS117614	AS117615	AS117616	AS117617	AS117618	AS117619
4" x .75" PLASTIC W/ POLY	CP103006	CP103006	AS117608	AS117609	AS117610	AS117611	AS117612	AS117613
4" x .75" VOLCANIC	CP103007	CP103007	AS117614	AS117615	AS117616	AS117617	AS117618	AS117619
4" x .75" LITESPEED W/ POLY	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
5" X 1" PLASTIC W/ POLY	CP103006	CP103006	AS117608	AS117609	AS117610	AS117611	AS117612	AS117613
5" x .75" LITESPEED W/ POLY	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
6" x 1" PLASTIC W/ POLY	CP103006	CP103006	AS117608	AS117609	AS117610	AS117611	AS117612	AS117613
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103006	CP103006	AS117608	AS117609	AS117610	AS117611	AS117612	AS117613
6" x .75" LITESPEED W/ POLY	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625
6" x 1.25" LITESPEED W/ PNEUMATIC	CP104403	CP104404	AS117620	AS117621	AS117622	AS117623	AS117624	AS117625



HEAVY DUTY BEARING FORK ASSEMBLY

TiLite™ Aero X, 2GX

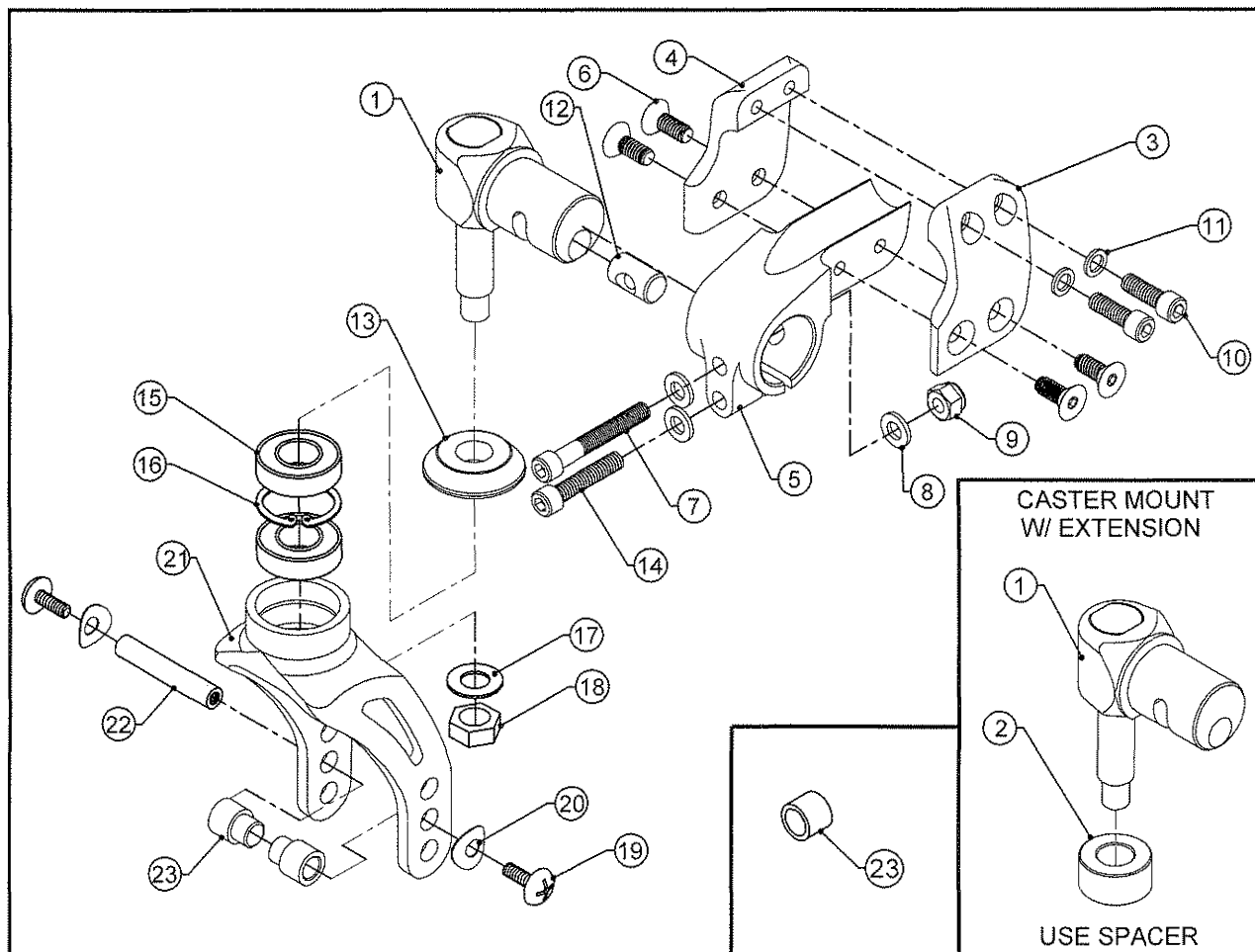

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS110375	HD BEARING FORK CASTER MOUNT ASSEMBLY
1B	1	AS110376	HD BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
1C	1	AS110377	HD BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
2A	1	CP104035	HD BEARING FORK STEM 1/2" SPACER
2B	1	CP104036	HD BEARING FORK STEM 1" SPACER
3A	1	CP104042	HD CASTER MOUNT CLAMP, C'BORED, RIGHT
3B	1	CP104043	HD CASTER MOUNT CLAMP, C'BORED, LEFT
4A	1	CP104040	HD CASTER MOUNT CLAMP, THREADED, RIGHT
4B	1	CP104041	HD CASTER MOUNT CLAMP, THREADED, LEFT
5	1	CP104039	HD CASTER MOUNT
6	4	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
7	1	HD100563	1/4-28 X 1-3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
8	3	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
9	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
10	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
11	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	1	CP102142	CASTER MOUNT DOWEL Ø.437
13	1	CP104037	HD BEARING FORK CAP
14	1	HD100562	1/4-20 X 1-1/4" SOCKET HEAD CAP SCREW, BLACK ZINC
15	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
16	1	HD100330	1-1/8" INTERNAL RETAINING RING
17	1	HD100353	3/8" AN WASHER STAINLESS STEEL
18	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
19	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
20	2	CP104375	C SPRING WASHER
21A	1	CP106957	WIDE BEARING FORK
21B	1	CP106956	WIDE BEARING FORK SHORT
21C	1	CP106958	BEARING FORK TALL
22	1	CP103041	CASTER AXLE LONG
23	2	TABLE ONE	CASTER SPACER

ASSEMBLIES ON NEXT PAGE

HEAVY DUTY BEARING FORK ASSEMBLY

TiLite™ Aero X, 2GX

ITEM	QTY	PART NO.	DESCRIPTION
1A,3A,4A-20,21A,22,23		TABLE ONE	HD BEARING FORK STD CASTER MOUNT ASSEMBLY, RIGHT
1A,3B,4B-20,21A,22,23		TABLE ONE	HD BEARING FORK STD CASTER MOUNT ASSEMBLY, LEFT
1B,2A,3A, 4A-20,21A,22,23		TABLE ONE	HD BEARING FORK STD CASTER MOUNT ASSEMBLY, RIGHT W/ 1/2" EXT
1B,2A,3B, 4B-20,21A,22,23		TABLE ONE	HD BEARING FORK STD CASTER MOUNT ASSEMBLY, LEFT W/ 1/2" EXT
1C,2B,3A,4A-20,21A,22,23		TABLE ONE	HD BEARING FORK STD CASTER MOUNT ASSEMBLY, RIGHT W/ 1" EXT
1C,2B,3B,4B-20,21A,22,23		TABLE ONE	HD BEARING FORK STD CASTER MOUNT ASSEMBLY, LEFT W/ 1" EXT
1A,3A,4A-20,21B,22,23		TABLE TWO	HD BEARING FORK SHORT CASTER MOUNT ASSEMBLY, RIGHT
1A,3B,4B-20,21B,22,23		TABLE TWO	HD BEARING FORK SHORT CASTER MOUNT ASSEMBLY, LEFT
1B,2A,3A,4A-20,21B,22,23		TABLE TWO	HD BEARING FORK SHORT CASTER MOUNT ASSEMBLY, RIGHT W/ 1/2" EXT
1B,2A,3B,4B-20,21B,22,23		TABLE TWO	HD BEARING FORK SHORT CASTER MOUNT ASSEMBLY, LEFT W/ 1/2" EXT
1C,2B,3A,4A-20,21B,22,23		TABLE TWO	HD BEARING FORK SHORT CASTER MOUNT ASSEMBLY, RIGHT W/ 1" EXT
1C,2B,3B,4B-20,21B,22,23		TABLE TWO	HD BEARING FORK SHORT CASTER MOUNT ASSEMBLY, LEFT W/ 1" EXT
1A,3A,4A-20,21C,22,23		TABLE THREE	HD BEARING FORK TALL CASTER MOUNT ASSEMBLY, RIGHT 3"-4" CASTER
1A,3B,4B-20,21C,22,23		TABLE THREE	HD BEARING FORK TALL CASTER MOUNT ASSEMBLY, LEFT 3"-4" CASTER
1B,2A,3A,4A-20,21C,22,23		TABLE THREE	HD BEARING FORK TALL CASTER MOUNT ASSEMBLY, RIGHT 3"-4" CASTER W/ 1/2" EXT
1B,2A,3B,4B-20,21C,22,23		TABLE THREE	HD BEARING FORK TALL CASTER MOUNT ASSEMBLY, LEFT 3"-4" CASTER W/ 1/2" EXT
1C,2B,3A,4A-20,21C,22,23		TABLE THREE	HD BEARING FORK TALL CASTER MOUNT ASSEMBLY, RIGHT 3"-4" CASTER W/ 1" EXT
1C,2B,3B,4B-20,21C,22,23		TABLE THREE	HD BEARING FORK TALL CASTER MOUNT ASSEMBLY, LEFT 3"-4" CASTER W/ 1" EXT

TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK RIGHT	STD FORK LEFT	STD FORK 1/2" EXT RIGHT	STD FORK 1/2" EXT LEFT	STD FORK 1" EXT RIGHT	STD FORK 1" EXT LEFT
3" MICRO	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383
3" VOLCANIC	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383
4" PLASTIC W/ POLY	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383
4" VOLCANIC	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383
4" LITESPEED W/ POLY	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
5" PLASTIC W/ POLY	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383
5" LITESPEED W/ POLY	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
6" PLASTIC W/ POLY	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383
6" LITESPEED W/ POLY	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111396	AS111397	AS111398	AS111399	AS111400	AS111401
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS110378	AS110379	AS110380	AS110381	AS110382	AS110383

TABLE TWO - SPACERS AND SHORT FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	SHORT FORK RIGHT	SHORT FORK LEFT	SHORT FORK 1/2" EXT RIGHT	SHORT FORK 1/2" EXT LEFT	SHORT FORK 1" EXT RIGHT	SHORT FORK 1" EXT LEFT
3" MICRO	CP103945	CP103945	AS110384	AS110385	AS110386	AS110387	AS110388	AS110389
3" VOLCANIC	CP103945	CP103945	AS110384	AS110385	AS110386	AS110387	AS110388	AS110389
4" PLASTIC W/ POLY	CP103945	CP103945	AS110384	AS110385	AS110386	AS110387	AS110388	AS110389
4" VOLCANIC	CP103945	CP103945	AS110384	AS110385	AS110386	AS110387	AS110388	AS110389
4" LITESPEED W/ POLY	CP104405	CP104406	AS111402	AS111403	AS111404	AS111405	AS111406	AS111407
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111402	AS111403	AS111404	AS111405	AS111406	AS111407
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111402	AS111403	AS111404	AS111405	AS111406	AS111407


TABLE THREE - SPACERS AND TALL FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	TALL FORK RIGHT	TALL FORK LEFT	TALL FORK 1/2" EXT RIGHT	TALL FORK 1/2" EXT LEFT	TALL FORK 1" EXT RIGHT	TALL FORK 1" EXT LEFT
3" MICRO	CP103007	CP103007	AS110390	AS110391	AS110394	AS110395	AS110398	AS110399
3" VOLCANIC	CP103007	CP103007	AS110390	AS110391	AS110394	AS110395	AS110398	AS110399
4" PLASTIC W/ POLY	CP103006	CP103006	AS110392	AS110393	AS110396	AS110397	AS110400	AS110401
4" VOLCANIC	CP103007	CP103007	AS110390	AS110391	AS110394	AS110395	AS110398	AS110399
4" LITESPEED W/ POLY	CP104403	CP104404	AS111408	AS111409	AS111410	AS111411	AS111412	AS111413
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111408	AS111409	AS111410	AS111411	AS111412	AS111413
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP103006	CP103006	AS110392	AS110393	AS110396	AS110397	AS110400	AS110401
5" PLASTIC W/ POLY	CP103006	CP103006	AS110392	AS110393	AS110396	AS110397	AS110400	AS110401
5" LITESPEED W/ POLY	CP104403	CP104404	AS111408	AS111409	AS111410	AS111411	AS111412	AS111413
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111408	AS111409	AS111410	AS111411	AS111412	AS111413
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP103006	CP103006	AS110392	AS110393	AS110396	AS110397	AS110400	AS110401
6" PLASTIC W/ POLY	CP103006	CP103006	AS110392	AS110393	AS110396	AS110397	AS110400	AS110401
6" PLASTIC W/ PNEUMATIC	CP103006	CP103006	AS110392	AS110393	AS110396	AS110397	AS110400	AS110401
6" LITESPEED W/ POLY	CP104403	CP104404	AS111408	AS111409	AS111410	AS111411	AS111412	AS111413
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111408	AS111409	AS111410	AS111411	AS111412	AS111413
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS111408	AS111409	AS111410	AS111411	AS111412	AS111413
6" LITESPEED W/ PNEUMATIC	CP103006	CP103006	AS110392	AS110393	AS110396	AS110397	AS110400	AS110401



WIDE BEARING FORK ASSEMBLY

HEMI HEIGHT TiLite™ Aero X

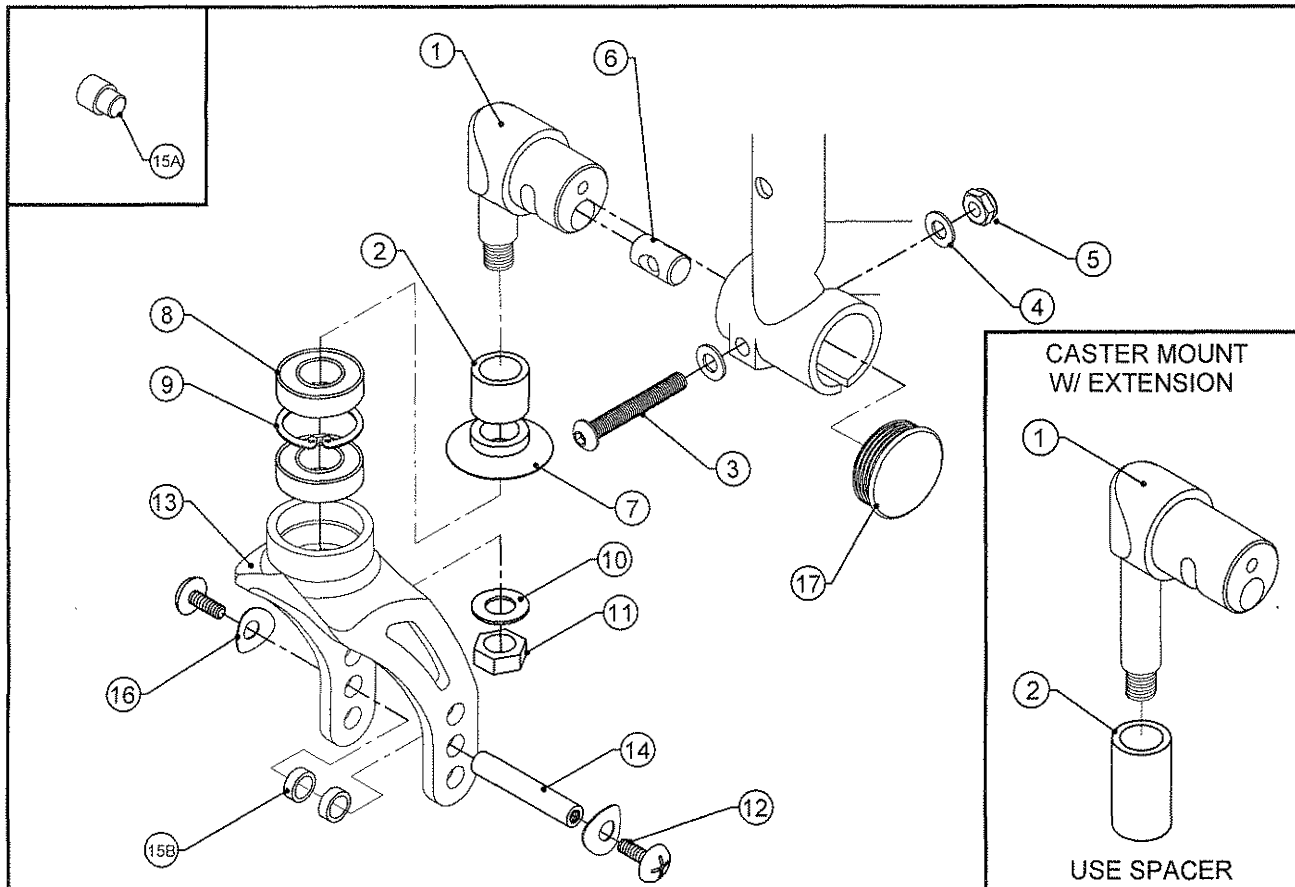

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS111159	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY
1B	1	AS111160	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
1C	1	AS111161	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
2A	1	CP100346	CASTER EXTENSION 1/2"
2B	1	CP100347	CASTER EXTENSION 1"
3	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
4	2	HD100402	WASHER 1/4" LIGHT AN (.265 ID X .500 OD X .032 THK)
5	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
6	1	CP102142	CASTER MOUNT DOWEL Ø.437
7	1	CP102048	CASTER CAP BEARING FORK
8	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
9	1	HD100330	1-1/8" INTERNAL RETAINING RING
10	1	HD100353	3/8" AN WASHER STAINLESS STEEL
11	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
12	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
13A	1	CP106957	WIDE BEARING FORK
13B	1	CP106956	WIDE BEARING FORK SHORT
13C	1	CP106958	BEARING FORK TALL
14	1	CP103041	CASTER AXLE LONG
15A	2	TABLE ONE	CASTER SPACERS
15B	2	CP103006	.316ID X .438OD X .317L CASTER AXLE SPACER
16	2	CP104375	C SPRING WASHER
17	1	CP102064	ENDCAP 1.25 OD X .125 WALL
1A,3-12,13B,14,15A,16,17		AS111162	HEMI HEIGHT WIDE BEARING FORK SHORT CASTER MOUNT
1B,2A,3-12,13B,14,15A,16,17		AS111163	HEMI HEIGHT WIDE BEARING FORK SHORT CASTER MOUNT W/ 1/2" EXT
1C,2B,3-12,13B,14,15A,16,17		AS111164	HEMI HEIGHT WIDE BEARING FORK SHORT CASTER MOUNT W/ 1" EXT
1A,3-12,13A,14,15A,16,17		AS111165	HEMI HEIGHT WIDE BEARING FORK STD CASTER MOUNT
1B,2A,3-12,13A,14,15A,16,17		AS111166	HEMI HEIGHT WIDE BEARING FORK STD CASTER MOUNT W/ 1/2" EXT
1C,2B,3-12,13A,14,15A,16,17		AS111167	HEMI HEIGHT WIDE BEARING FORK STD CASTER MOUNT W/ 1" EXT
1A,3-12,13C,14,15B,16,17		AS111168	HEMI HEIGHT WIDE BEARING FORK TALL CASTER MOUNT
1B,2A,3-12,13C,14,15B,16,17		AS111169	HEMI HEIGHT WIDE BEARING FORK TALL CASTER MOUNT W/ 1/2" EXT
1C,2B,3-12,13C,14,15B,16,17		AS111170	HEMI HEIGHT WIDE BEARING FORK TALL CASTER MOUNT W/ 1" EXT

WIDE BEARING FORK ASSEMBLY

HEMI HEIGHT TiLite™ Aero X


TABLE ONE - CASTER SPACERS

	INSIDE SPACER	OUTSIDE SPACER
3" x .75" MICRO	CP103945	CP103945
3" x .75" VOLCANIC	CP103945	CP103945
4" x .75" PLASTIC W/ POLY	CP103945	CP103945
4" x .75" VOLCANIC	CP103945	CP103945
4" x .75" LITESPEED W/ POLY	CP104405	CP104406
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406
5" x 1" PLASTIC W/ POLY	CP103945	CP103945
5" x .75" LITESPEED W/ POLY	CP104405	CP104406
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406
6" x 1" PLASTIC W/ POLY	CP103945	CP103945
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103945	CP103945
6" x .75" LITESPEED W/ POLY	CP104405	CP104406
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406
6" x 1.25" LITESPEED W/ PNEUMATIC	CP103945	CP103945



WIDE BEARING FORK ASSEMBLY

HEMI HEIGHT TiLite™ 2GX

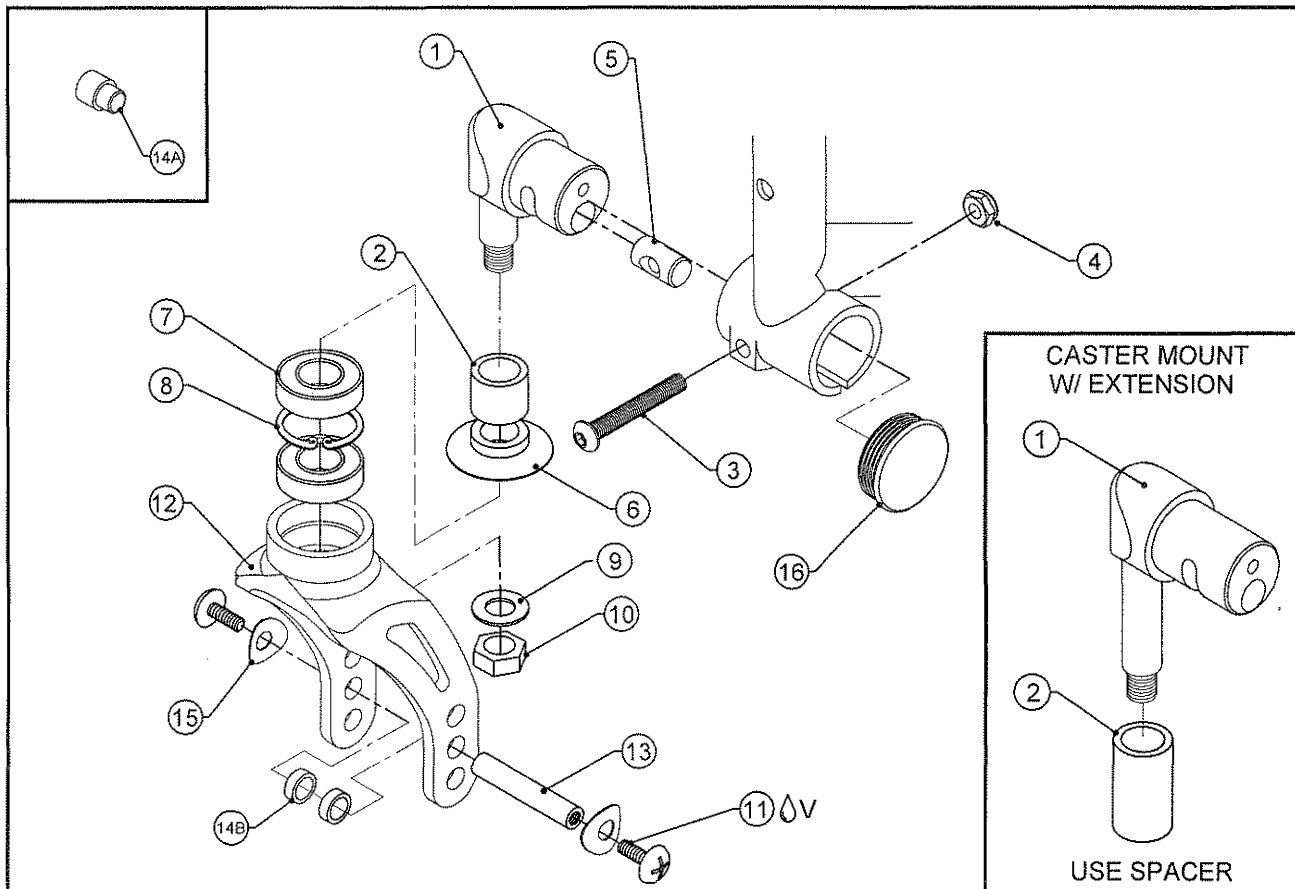


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS111159	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY
1B	1	AS111160	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
1C	1	AS111161	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
2A	1	CP100346	CASTER EXTENSION 1/2"
2B	1	CP100347	CASTER EXTENSION 1"
3	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
4	1	HD100333	1/4-28 NYLOCK NUT THIN STYLE, BLACK ZINC
5	1	CP102142	CASTER MOUNT DOWEL Ø.437
6	1	CP102048	CASTER CAP BEARING FORK
7	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
8	1	HD100330	1-1/8" INTERNAL RETAINING RING
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
11	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
12A	1	CP106957	WIDE BEARING FORK
12B	1	CP106956	WIDE BEARING FORK SHORT
12C	1	CP106958	BEARING FORK TALL
13	1	CP103041	CASTER AXLE LONG
14A	2	TABLE ONE	CASTER SPACERS
14B	2	CP103006	.316ID X .438OD X .317L CASTER AXLE SPACER
15	2	CP104375	C SPRING WASHER
16	1	CP102064	ENDCAP 1.25 OD X .125 WALL
1A,3-11,12B,13,14A,15,16		AS115926	HEMI HEIGHT WIDE BEARING FORK SHORT CASTER MOUNT
1B,2A,3-11,12B,13,14A,15,16		AS115927	HEMI HEIGHT WIDE BEARING FORK SHORT CASTER MOUNT W/ 1/2" EXT
1C,2B,3-11,12B,13,14A,15,16		AS115928	HEMI HEIGHT WIDE BEARING FORK SHORT CASTER MOUNT W/ 1" EXT
1A,3-11,12A,13,14A,15,16		AS115929	HEMI HEIGHT WIDE BEARING FORK STD CASTER MOUNT
1B,2A,3-11,12A,13,14A,15,16		AS115930	HEMI HEIGHT WIDE BEARING FORK STD CASTER MOUNT W/ 1/2" EXT
1C,2B,3-11,12A,13,14A,15,16		AS115931	HEMI HEIGHT WIDE BEARING FORK STD CASTER MOUNT W/ 1" EXT
1A,3-11,12C,13,14B,15,16		AS115932	HEMI HEIGHT WIDE BEARING FORK TALL CASTER MOUNT
1B,2A,3-11,12C,13,14B,15,16		AS115933	HEMI HEIGHT WIDE BEARING FORK TALL CASTER MOUNT W/ 1/2" EXT
1C,2B,3-11,12C,13,14B,15,16		AS115934	HEMI HEIGHT WIDE BEARING FORK TALL CASTER MOUNT W/ 1" EXT

WIDE BEARING FORK ASSEMBLY

HEMI HEIGHT TiLite™ 2GX


TABLE ONE - CASTER SPACERS

	INSIDE SPACER	OUTSIDE SPACER
3" x .75" MICRO	CP103945	CP103945
3" x .75" VOLCANIC	CP103945	CP103945
4" x .75" PLASTIC W/ POLY	CP103945	CP103945
4" x .75" VOLCANIC	CP103945	CP103945
4" x .75" LITESPEED W/ POLY	CP104405	CP104406
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406
5" x 1" PLASTIC W/ POLY	CP103945	CP103945
5" x .75" LITESPEED W/ POLY	CP104405	CP104406
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406
6" x 1" PLASTIC W/ POLY	CP103945	CP103945
6" x 1.25" PLASTIC W/ PNEUMATIC	CP103945	CP103945
6" x .75" LITESPEED W/ POLY	CP104405	CP104406
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406
6" x 1.25" LITESPEED W/ PNEUMATIC	CP103945	CP103945

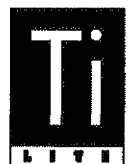

BEARING FORK ASSEMBLY


TiLite™ ZR, TR(SERIES 2), TRC(SERIES 2), YR(SERIES 2), TX

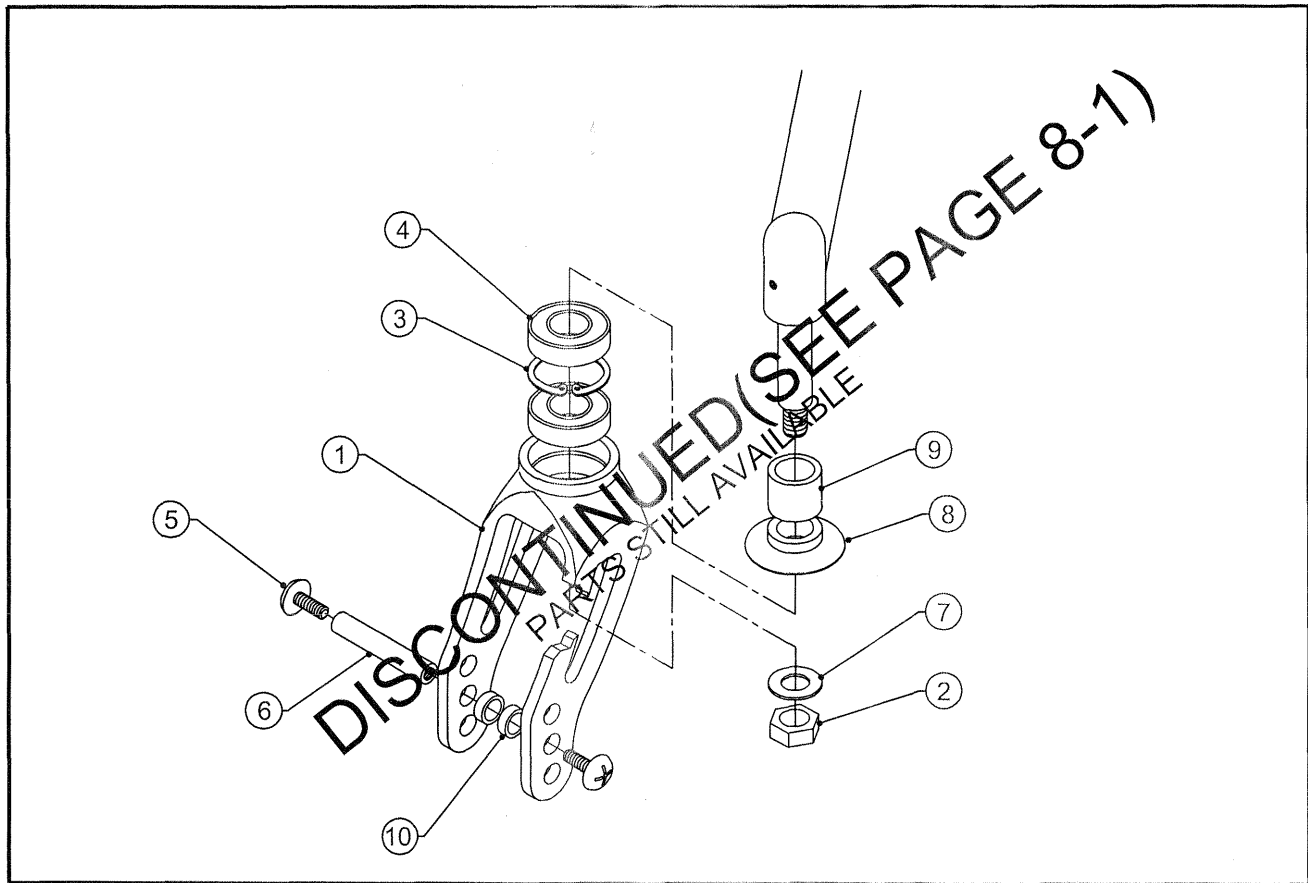

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP102049	BEARING FORK
2	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
3	1	HD100330	1-1/8" INTERNAL RETAINING RING
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
6	1	HD100085	5"-8" CASTER AXLE
7	1	HD100353	3/8" AN WASHER STAINLESS STEEL
8	1	CP102048	CASTER CAP BEARING FORK
9	1,2	CP102093	ZRA CASTER EXTENSION 1/2"
10A	2	HD100084	438 X .220 SPACER FOR 5" CASTER
10B	2	CP100007	FORK SPACER
1-8,10B		AS102811	ZR BEARING FORK ASSEMBLY FOR 3"-4" CASTER
1-8,10A		AS102812	ZR BEARING FORK ASSEMBLY FOR 5" CASTER
1-9,10B		AS102813	ZR BEARING FORK ASSEMBLY FOR 3"-4" CASTER W/ 1/2" EXT
1-9,10B		AS102814	ZR BEARING FORK ASSEMBLY FOR 3"-4" CASTER W/ 1" EXT
1-9,10A		AS102815	ZR BEARING FORK ASSEMBLY FOR 5" CASTER W/ 1/2" EXT
1-9,10A		AS102816	ZR BEARING FORK ASSEMBLY FOR 5" CASTER W/ 1" EXT
10,11B		AS102361	ZRA 1/2" CASTER EXTENSION PACKAGE
10,11C		AS102362	ZRA 1" CASTER EXTENSION PACKAGE


BEARING FORK ASSEMBLY

TiLite™ ZRA

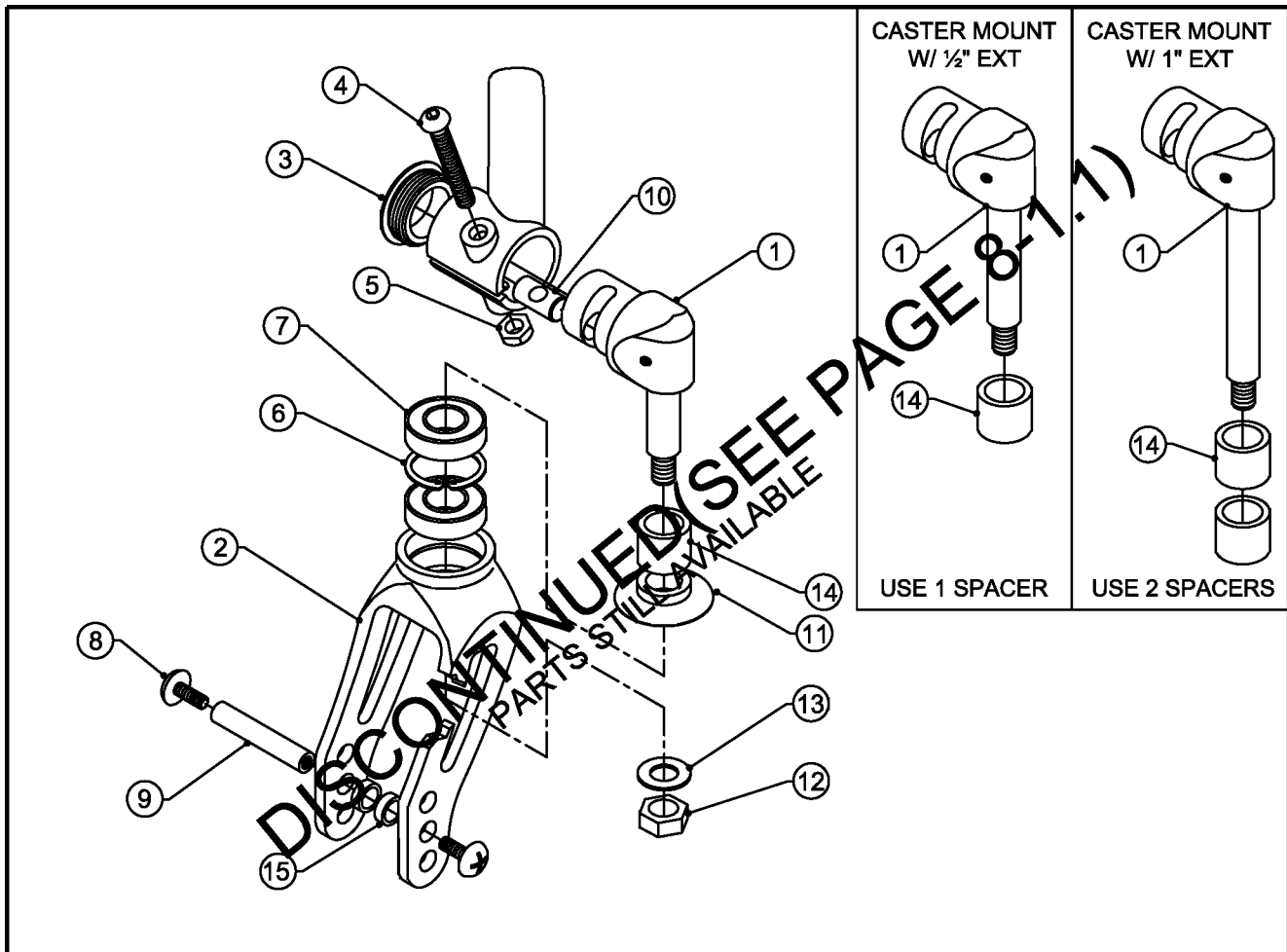

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS102842	CASTER MOUNT FOR BEARING FORK RIGHT ASSEMBLY
1B	1	AS102843	CASTER MOUNT FOR BEARING FORK LEFT ASSEMBLY
1C	1	AS102844	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT RIGHT ASSEMBLY
1D	1	AS102845	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT LEFT ASSEMBLY
1E	1	AS102846	CASTER MOUNT FOR BEARING FORK W/ 1" EXT RIGHT ASSEMBLY
1F	1	AS102847	CASTER MOUNT FOR BEARING FORK W/ 1" EXT LEFT ASSEMBLY
2	1	CP102049	BEARING FORK
3	1	CP102064	ENDCAP 1.25 OD X .125 WALL
4	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
5	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
6	1	HD100330	1-1/8" INTERNAL RETAINING RING
7	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
8	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
9	1	HD100085	5"-8" CASTER AXLE
10	1	CP102142	CASTER MOUNT DOWEL Ø.437
11	1	CP102048	CASTER CAP BEARING FORK
12	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
13	1	HD100353	3/8" AN WASHER STAINLESS STEEL
14	1,2	CP102093	ZRA CASTER EXTENSION 1/2"
15A	2	HD100084	438 X .220 SPACER FOR 5" CASTER
15B	2	CP100007	FORK SPACER

ASSEMBLIES ON NEXT PAGE

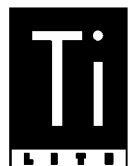


BEARING FORK ASSEMBLY (CONT.)

TiLite™ ZRA


ITEM	QTY	PART NO.	DESCRIPTION
1B-13,15B		AS102265	ZRA BEARING FORK ASSEMBLY LEFT FOR 3"-4" CASTER
1A-13,15B		AS102266	ZRA BEARING FORK ASSEMBLY RIGHT FOR 3"-4" CASTER
1B-13,15A		AS102269	ZRA BEARING FORK ASSEMBLY LEFT FOR 5" CASTER
1A-13,15A		AS102270	ZRA BEARING FORK ASSEMBLY RIGHT FOR 5" CASTER
1D-14,15B		AS102280	ZRA BEARING FORK ASSEMBLY LEFT FOR 3"-4" CASTER W/ 1/2" EXT
1C-14,15B		AS102281	ZRA BEARING FORK ASSEMBLY RIGHT FOR 3"-4" CASTER W/ 1/2" EXT
1F-14,15B		AS102282	ZRA BEARING FORK ASSEMBLY LEFT FOR 3"-4" CASTER W/ 1" EXT
1E-14,15B		AS102283	ZRA BEARING FORK ASSEMBLY RIGHT FOR 3"-4" CASTER W/ 1" EXT
1D-14,15A		AS102284	ZRA BEARING FORK ASSEMBLY LEFT FOR 5" CASTER W/ 1/2" EXT
1C-14,15A		AS102285	ZRA BEARING FORK ASSEMBLY RIGHT FOR 5" CASTER W/ 1/2" EXT
1F-14,15A		AS102286	ZRA BEARING FORK ASSEMBLY LEFT FOR 5" CASTER W/ 1" EXT
1E-14,15A		AS102287	ZRA BEARING FORK ASSEMBLY RIGHT FOR 5" CASTER W/ 1" EXT

DISCONTINUED(SEE PAGE 8-1.2)
PARTS STILL AVAILABLE

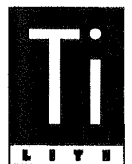

BEARING FORK ASSEMBLY

TiLite™ AERO Z

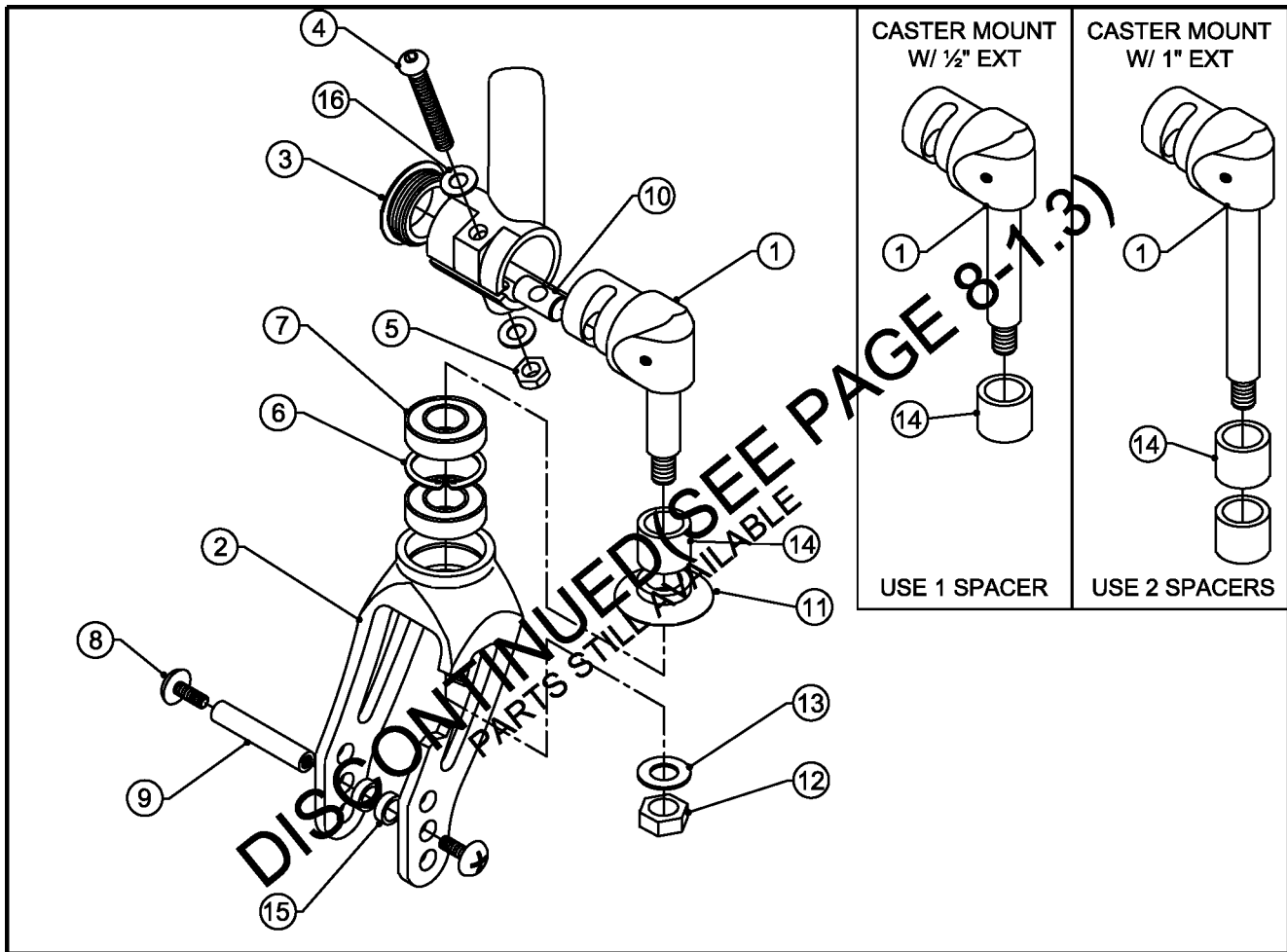

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS102842	CASTER MOUNT FOR BEARING FORK RIGHT ASSEMBLY
1B	1	AS102843	CASTER MOUNT FOR BEARING FORK LEFT ASSEMBLY
1C	1	AS102844	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT RIGHT ASSEMBLY
1D	1	AS102845	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT LEFT ASSEMBLY
1E	1	AS102846	CASTER MOUNT FOR BEARING FORK W/ 1" EXT RIGHT ASSEMBLY
1F	1	AS102847	CASTER MOUNT FOR BEARING FORK W/ 1" EXT LEFT ASSEMBLY
2	1	CP102049	BEARING FORK
3	1	CP102064	ENDCAP 1.25 OD X .125 WALL
4	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
5	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
6	1	HD100330	1-1/8" INTERNAL RETAINING RING
7	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
8	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
9	1	HD100085	5"-8" CASTER AXLE
10	1	CP102142	CASTER MOUNT DOWEL Ø.437
11	1	CP102048	CASTER CAP BEARING FORK
12	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
13	1	HD100353	3/8" AN WASHER STAINLESS STEEL
14	1,2	CP102093	ZRA CASTER EXTENSION 1/2"
15A	2	HD100084	438 X .220 SPACER FOR 5" CASTER
15B	2	CP100007	FORK SPACER
16	2	HD100402	WASHER 1/4" LIGHT AN (.265 ID X .500 OD X .032 THK)

ASSEMBLIES ON NEXT PAGE

BEARING FORK ASSEMBLY (CONT.)

TiLite™ AERO Z


ITEM	QTY	PART NO.	DESCRIPTION
1B-13,15B,16		AS103254	AERO Z BEARING FORK ASSEMBLY LEFT FOR 3"-4" CASTER
1A-13,15B,16		AS103255	AERO Z BEARING FORK ASSEMBLY RIGHT FOR 3"-4" CASTER
1B-13,15A,16		AS103256	AERO Z BEARING FORK ASSEMBLY LEFT FOR 5" CASTER
1A-13,15A,16		AS103257	AERO Z BEARING FORK ASSEMBLY RIGHT FOR 5" CASTER
1D-14,15B,16		AS103258	AERO Z BEARING FORK ASSEMBLY LEFT FOR 3"-4" CASTER W/ 1/2" EXT
1C-14,15B,16		AS103259	AERO Z BEARING FORK ASSEMBLY RIGHT FOR 3"-4" CASTER W/ 1/2" EXT
1D-14,15A,16		AS103260	AERO Z BEARING FORK ASSEMBLY LEFT FOR 5" CASTER W/ 1/2" EXT
1C-14,15A,16		AS103261	AERO Z BEARING FORK ASSEMBLY RIGHT FOR 5" CASTER W/ 1/2" EXT
1F-14,15B,16		AS103262	AERO Z BEARING FORK ASSEMBLY LEFT FOR 3"-4" CASTER W/ 1" EXT
1E-14,15B,16		AS103263	AERO Z BEARING FORK ASSEMBLY RIGHT FOR 3"-4" CASTER W/ 1" EXT
1F-14,15A,16		AS103264	AERO Z BEARING FORK ASSEMBLY LEFT FOR 5" CASTER W/ 1" EXT
1E-14,15A,16		AS103265	AERO Z BEARING FORK ASSEMBLY RIGHT FOR 5" CASTER W/ 1" EXT

DISCONTINUED(SEE PAGE 8-1.4)
PARTS STILL AVAILABLE



BEARING FORK ASSEMBLY

TiLite™ Aero X, 2GX, Aero R

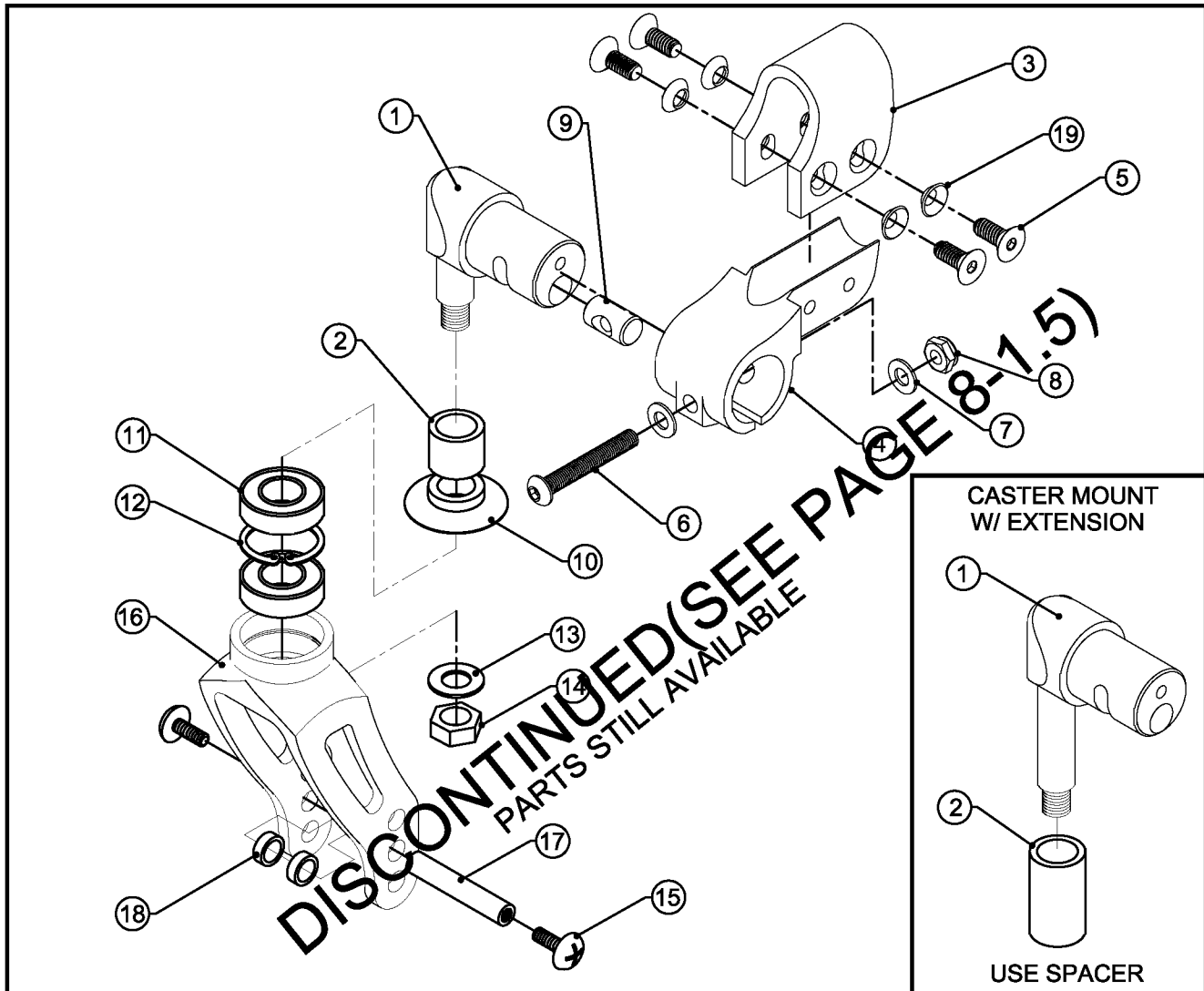

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS103237	BEARING FORK CASTER MOUNT ASSEMBLY
1B	1	AS103238	BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
1C	1	AS103239	BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
2A	1	CP100346	CASTER EXTENSION 1/2"
2B	1	CP100347	CASTER EXTENSION 1"
3	1	CP103186	CASTER MOUNT TOP
4	1	CP103185	CASTER MOUNT
5	4	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
6	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
7	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
8	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
9	1	CP102142	CASTER MOUNT DOWEL Ø.437
10	1	CP102048	CASTER CAP BEARING FORK
11	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
12	1	HD100330	1-1/8" INTERNAL RETAINING RING
13	1	HD100353	3/8" AN WASHER STAINLESS STEEL
14	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
15	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
16A	1	CP102049	BEARING FORK
16B	1	CP103040	BEARING FORK SHORT
16C	1	CP103039	BEARING FORK TALL
17A	1	HD100085	5"-8" CASTER AXLE
17B	1	CP103041	CASTER AXLE LONG
18A	2	HD100084	.438 X .220 SPACER FOR 5" CASTER
18B	2	CP100007	FORK SPACER
18C	2	CP103006	.316ID X .438OD X .317L CASTER AXLE SPACER
18D	2	CP103007	.316ID X .438OD X .338L CASTER AXLE SPACER
19	4	HD100526	1/4" COUNTERSUNK WASHER

ASSEMBLIES ON NEXT PAGE
8-2.5

BEARING FORK ASSEMBLY

TiLite™ Aero X, 2GX, Aero R


ITEM	QTY	PART NO.	DESCRIPTION
1A,3-16A,17A,18B,19		AS103464	BEARING FORK CASTER MOUNT, 3"-4" CASTER
1A,3-16A,17A,18A,19		AS103465	BEARING FORK CASTER MOUNT, 5" CASTER
1B,2A,3-16A,17A,18B,19		AS103466	BEARING FORK CASTER MOUNT, 3"-4" CASTER, 1/2" EXT
1B,2A,3-16A,17A,18A,19		AS103467	BEARING FORK CASTER MOUNT, 5" CASTER, 1/2" EXT
1C,2B,3-16A,17A,18B,19		AS103468	BEARING FORK CASTER MOUNT, 3"-4" CASTER, 1" EXT
1C,2B,3-16A,17A,18A,19		AS103469	BEARING FORK CASTER MOUNT, 5" CASTER, 1" EXT
1A,3-16B,17A,18B,19		AS103470	BEARING FORK SHORT CASTER MOUNT, 3"-4" CASTER
1B,2A,3-16B,17A,18B,19		AS103471	BEARING FORK SHORT CASTER MOUNT, 3"-4" CASTER, 1/2" EXT
1C,2B,3-16B,17A,18B,19		AS103472	BEARING FORK SHORT CASTER MOUNT, 3"-4" CASTER, 1" EXT
1A,3-16C,17B,18D,19		AS103473	BEARING FORK TALL CASTER MOUNT, 3"-4" CASTER
1A,3-16C,17B,18C,19		AS103474	BEARING FORK TALL CASTER MOUNT, 5" CASTER
1B,2A,3-16C,17B,18D,19		AS103475	BEARING FORK TALL CASTER MOUNT, 3"-4" CASTER, 1/2" EXT
1B,2A,3-16C,17B,18C,19		AS103476	BEARING FORK TALL CASTER MOUNT, 5" CASTER, 1/2" EXT
1C,2B,3-16C,17B,18D,19		AS103477	BEARING FORK TALL CASTER MOUNT, 3"-4" CASTER, 1" EXT
1C,2B,3-16C,17B,18C,19		AS103478	BEARING FORK TALL CASTER MOUNT, 5" CASTER, 1" EXT

DISCONTINUED (SEE PAGE 8-1.6)
PARTS STILL AVAILABLE



MONO FORK - CASTER ASSEMBLY

TiLite™ Z10

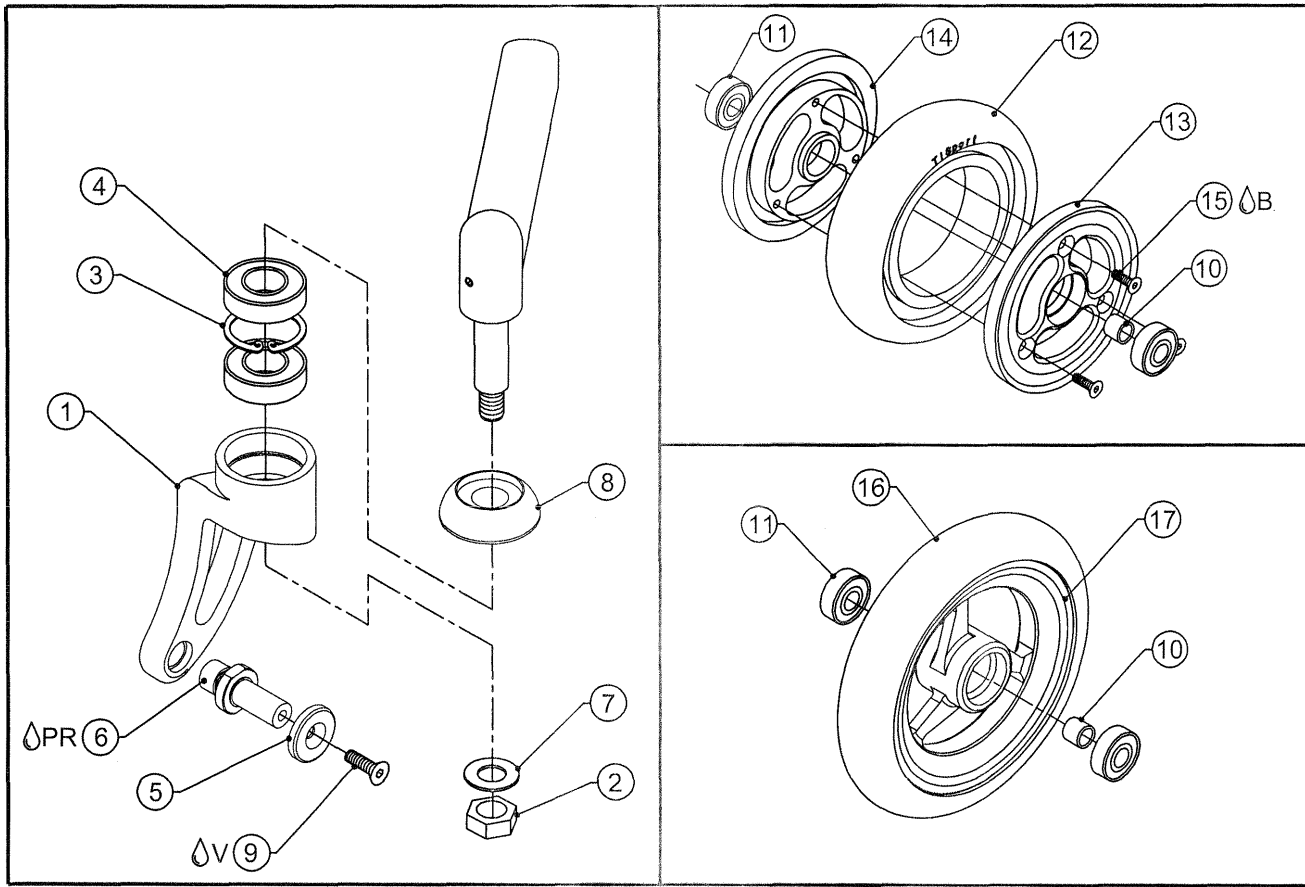


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP103746	MONO FORK, LEFT
1B	1	CP103747	MONO FORK, RIGHT
2	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
3	1	HD100330	1-1/8" INTERNAL RETAINING RING
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5A	1	CP104053	MONOFORK CAP CLEAR
5B	1	CP104054	MONOFORK CAP BLACK
6 ΔPR	1	CP103745	AXLE, MONO FORK
7	1	HD100353	3/8" AN WASHER STAINLESS STEEL
8	1	CP103744	CASTER CAP, MONO FORK
9A ΔV	1	HD100103	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, CLEAR ZINC
9B ΔV	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
10	1	CP103749	SPACER, MONO FORK HUB
11	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED
12	1	CP101303	4" SOFT ROLL TIRE
13A	1	CP103739	4" MONO FORK SOFT ROLL HUB A
13B	1	CP103741	4" BLACK ANODIZE MONO FORK SOFT ROLL HUB A
14A	1	CP103740	4" MONO FORK SOFT ROLL HUB B
14B	1	CP103742	4" BLACK ANODIZE MONO FORK SOFT ROLL HUB B
15A ΔB	3	HD100278	8-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, CLEAR ZINC
15B ΔB	3	HD100513	8-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
16	1	CP100759	4" HIGH RESILIENT BLACK TIRE
17A	1	CP103737	4" MONO FORK WHEEL HUB, CLEAR
17B	1	CP103738	4" MONO FORK WHEEL HUB, BLACK
1A-4,5A,6-9A		AS104247	MONO FORK ASSEMBLY, LEFT
1A-4,5B,6-9B		AS104784	MONO FORK ASSEMBLY, LEFT BLACK
1B-4,5A,6-9A		AS104248	MONO FORK ASSEMBLY, RIGHT
1B-4,5B,6-9B		AS104785	MONO FORK ASSEMBLY, RIGHT BLACK
10-12,13A,14A,15A		AS104249	MONO FORK 4" SOFTROLL
10-12,13B,14B,15B		AS104250	MONO FORK 4" SOFTROLL, BLACK
10-11,16A,17A		AS104251	MONO FORK 4" POLY
10-11,16B,17B		AS104252	MONO FORK 4" POLY, BLACK



Parts Manual

CASTER PINLOCKS FOR FIXED CASTERS

TiLite™ ZR, TR(SERIES 2), TX

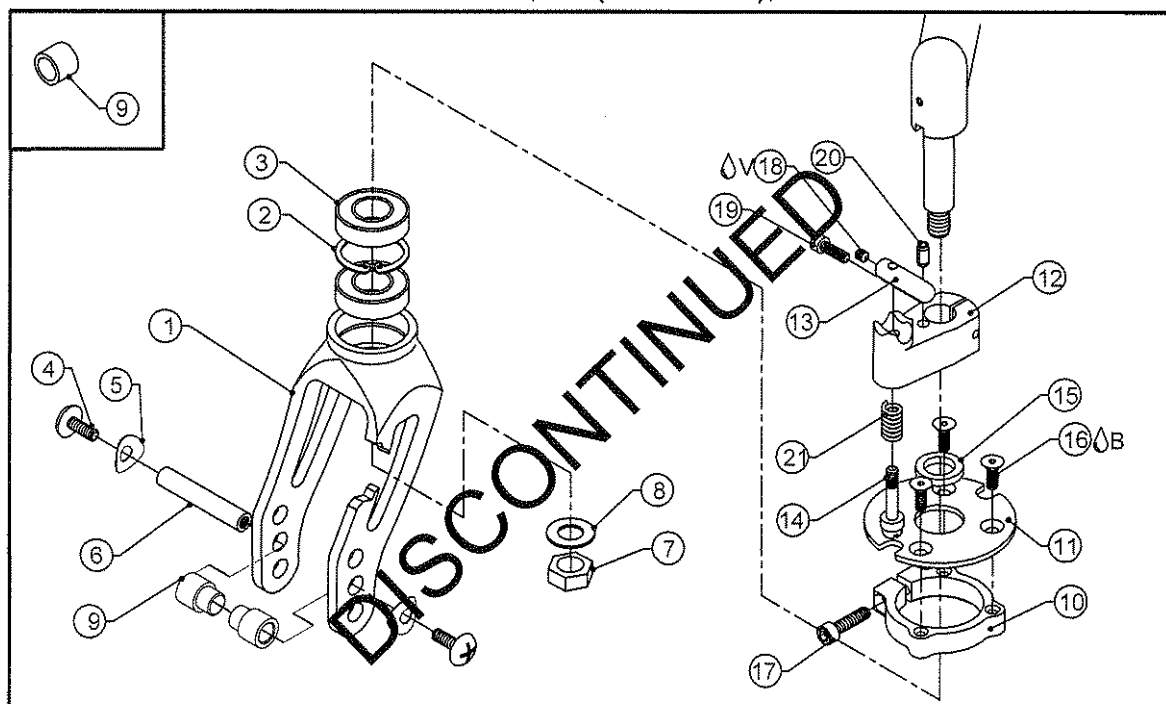

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	2	CP103566	WIDE BEARING FORK
2	2	HD100330	1-1/8" INTERNAL RETAINING RING
3	4	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
4	4	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
5	4	CP104375	C SPRING WASHER
6	2	HD100085	5"-8" CASTER AXLE
7	2	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
8	2	HD100353	3/8" AN WASHER STAINLESS STEEL
9	4	TABLE ONE	CASTER SPACER
10	2	CP102579	CASTER PINLOCK CLAMP
11	2	CP102581	CASTER PINLOCK PLATE, STEEL
12	2	CP102580	CASTER PINLOCK MOUNT B
13	2	CP102582	CASTER PINLOCK RELEASE HANDLE
14	2	CP102583	CASTER PINLOCK PIN
15	2	CP102584	SPACER ALUMINUM .500ID X .750OD X .125THICK
16B	2	HD100426	8-32 X 1/2" FHCS, STAINLESS STEEL
17	2	HD100427	10-32 X 3/4" SHCS, BLACK ZINC
18V	2	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
19	2	HD100425	8-32 X 1/2" SHCS, BLACK ZINC
20	2	HD100158	3/16" X 3/8" ROLL PIN
21	2	HD100429	SPRING, .328 OD. X .278 ID. X .630 LONG
1-21		AS103167	CASTER PINLOCK FOR FIXED CASTER ASSY, PAIR


TABLE ONE - SPACERS AND STD FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK
3" MICRO	CP103945	CP103945	AS103167
3" VOLCANIC	CP103945	CP103945	AS103167
4" PLASTIC W/ POLY	CP103945	CP103945	AS103167
4" VOLCANIC	CP103945	CP103945	AS103167
4" LITESPEED W/ POLY	CP104405	CP104406	AS111414
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111414
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP103945	CP103945	AS103167
5" PLASTIC W/ POLY	CP103945	CP103945	AS103167
5" LITESPEED W/ POLY	CP104405	CP104406	AS111414
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111414
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111414
6" PLASTIC W/ POLY	CP103945	CP103945	AS103167
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS103167
6" LITESPEED W/ POLY	CP104405	CP104406	AS111414
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111414
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111414
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS103167



CASTER PINLOCKS FOR BEARING FORK

TiLite™ ZRA, AERO Z

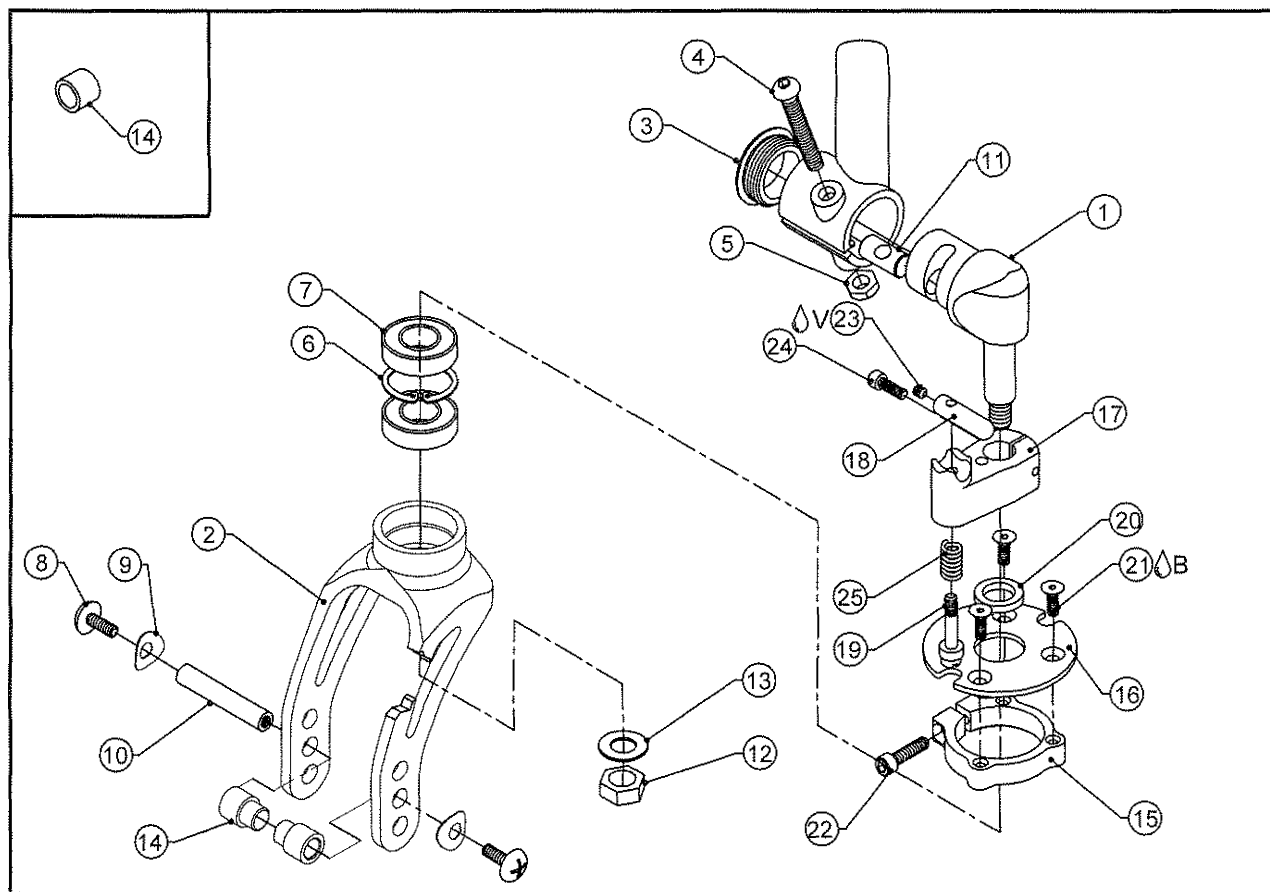

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	AS102844	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT RIGHT ASSEMBLY
1B	1	AS102845	CASTER MOUNT FOR BEARING FORK W/ 1/2" EXT LEFT ASSEMBLY
2A	1	CP106957	WIDE BEARING FORK
2B	1	CP106956	WIDE BEARING FORK SHORT
3	1	CP102064	ENDCAP 1.25 OD X .125 WALL
4	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
5	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
6	1	HD100330	1-1/8" INTERNAL RETAINING RING
7	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
8	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
9	2	CP104375	C SPRING WASHER
10	1	CP103041	CASTER AXLE LONG
11	1	CP102142	CASTER MOUNT DOWEL Ø.437
12	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
13	1	HD100353	3/8" AN WASHER STAINLESS STEEL
14	2	TABLE ONE	CASTER SPACER
15	1	CP102579	CASTER PINLOCK CLAMP
16	1	CP102581	CASTER PINLOCK PLATE, STEEL
17	1	CP102580	CASTER PINLOCK MOUNT
18	1	CP102582	CASTER PINLOCK RELEASE HANDLE
19	1	CP102583	CASTER PINLOCK PIN
20	1	CP102584	SPACER ALUMINUM .500ID X .750OD X .125THICK
21 ^Δ B	3	HD100426	8-32 X 1/2" FHCS, STAINLESS STEEL
22	1	HD100427	10-32 X 3/4" SHCS, BLACK ZINC
23 ^Δ V	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
24	1	HD100425	8-32 X 1/2" SHCS, BLACK ZINC
25	1	HD100429	SPRING, .328 OD. X .278 ID. X .630 LONG

ASSEMBLIES ON NEXT PAGE



Parts Manual

CASTER PINLOCKS FOR BEARING FORK

TiLite™ ZRA, AERO Z

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A,2A-12-25	TABLE ONE		CASTER PINLOCK WIDE BEARING FORK ASSY RIGHT
1B,2A-12-25	TABLE ONE		CASTER PINLOCK WIDE BEARING FORK ASSY LEFT
1A,1B,2A-12-25	TABLE ONE		CASTER PINLOCK WIDE BEARING FORK ASSY PAIR
1A,2B-12-25	TABLE ONE		CASTER PINLOCK WIDE SHORT BEARING FORK ASSY RIGHT
1B,2B-12-25	TABLE ONE		CASTER PINLOCK WIDE SHORT BEARING FORK ASSY LEFT
1A,1B,2B-12-25	TABLE ONE		CASTER PINLOCK WIDE SHORT BEARING FORK ASSY PAIR


TABLE ONE - SPACERS AND FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	STD FORK RIGHT	STD FORK LEFT	STD FORK PAIR	SHORT FORK RIGHT	SHORT FORK LEFT	SHORT FORK PAIR
3" MICRO	CP103945	CP103945	AS104628	AS104629	AS104630	AS104631	AS104632	AS104633
3" VOLCANIC	CP103945	CP103945	AS104628	AS104629	AS104630	AS104631	AS104632	AS104633
4" PLASTIC W/ POLY	CP103945	CP103945	AS104628	AS104629	AS104630	AS104631	AS104632	AS104633
4" VOLCANIC	CP103945	CP103945	AS104628	AS104629	AS104630	AS104631	AS104632	AS104633
4" LITESPEED W/ POLY	CP104405	CP104406	AS111415	AS111416	AS111419	AS111417	AS111418	AS111420
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111415	AS111416	AS111419	AS111417	AS111418	AS111420
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111415	AS111416	AS111419	AS111417	AS111418	AS111420
5" PLASTIC W/ POLY	CP103945	CP103945	AS104628	AS104629	AS104630	N/A	N/A	N/A
5" LITESPEED W/ POLY	CP104405	CP104406	AS111415	AS111416	AS111419	N/A	N/A	N/A
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111415	AS111416	AS111419	N/A	N/A	N/A
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111415	AS111416	AS111419	N/A	N/A	N/A
6" PLASTIC W/ POLY	CP103945	CP103945	AS104628	AS104629	AS104630	N/A	N/A	N/A
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS104628	AS104629	AS104630	N/A	N/A	N/A
6" LITESPEED W/ POLY	CP104405	CP104406	AS111415	AS111416	AS111419	N/A	N/A	N/A
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111415	AS111416	AS111419	N/A	N/A	N/A
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111415	AS111416	AS111419	N/A	N/A	N/A
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS104628	AS104629	AS104630	N/A	N/A	N/A



CASTER PINLOCKS FOR BEARING FORK

TiLite™ Aero X, 2GX, Aero R

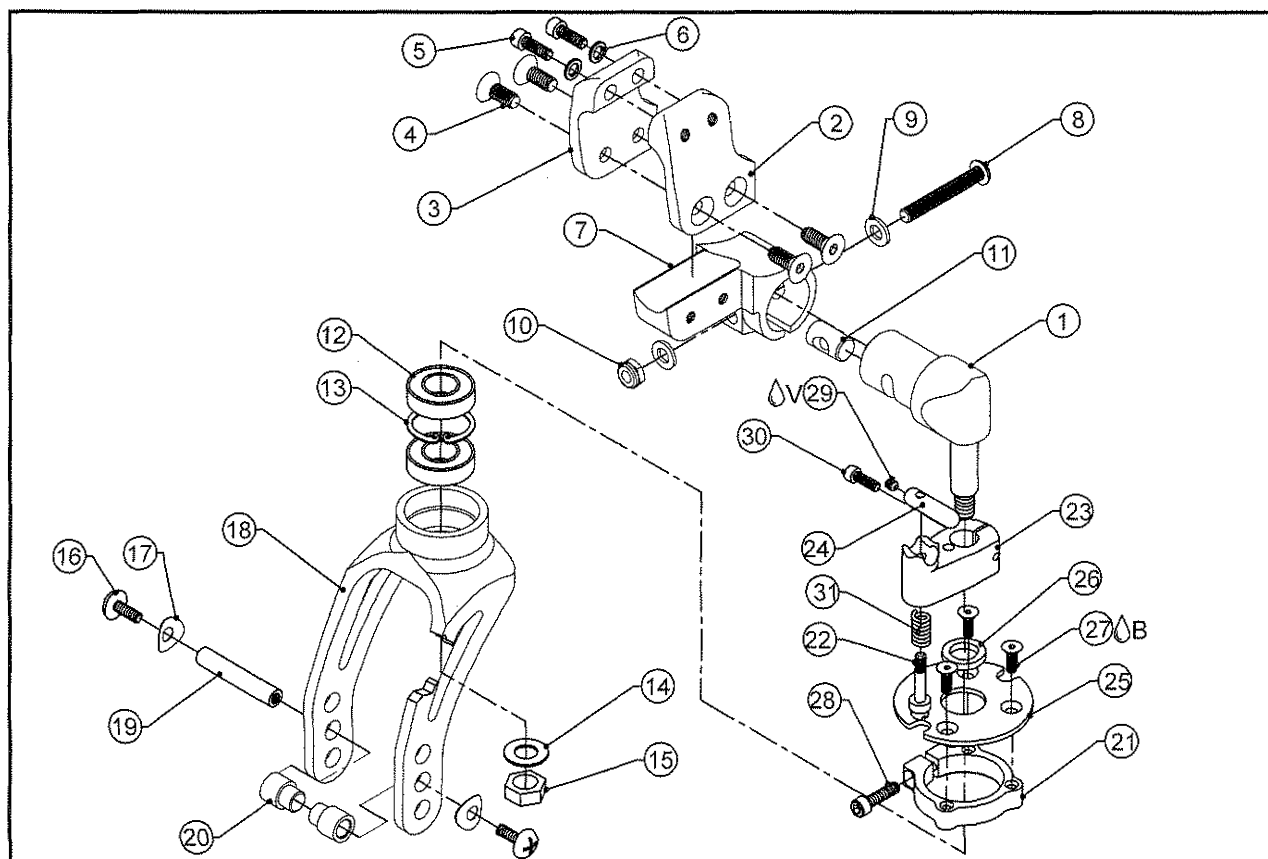

ITEM	QTY	PART NO.	DESCRIPTION
1	1	AS103238	BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
2A	1	CP103895	CASTER MOUNT CLAMP, TAPPED, RIGHT
2B	1	CP103897	CASTER MOUNT CLAMP, TAPPED, LEFT
3A	1	CP103898	CASTER MOUNT CLAMP, COUNTERBORED, RIGHT
3B	1	CP103896	CASTER MOUNT CLAMP, COUNTERBORED, LEFT
4	4	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
5	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
6	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, BLACK ZINC PLATED
7	1	CP103185	CASTER MOUNT
8	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
9	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
10	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
11	1	CP102142	CASTER MOUNT DOWEL Ø.437
12	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
13	1	HD100330	1-1/8" INTERNAL RETAINING RING
14	1	HD100353	3/8" AN WASHER STAINLESS STEEL
15	1	HD100352	3/8-24 NYLOCK NUT JAM HEIGHT
16	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
17	2	CP104375	C SPRING WASHER
18A	1	CP106957	WIDE BEARING FORK
18B	1	CP106956	WIDE BEARING FORK SHORT
18C	1	CP106958	WIDE BEARING FORK TALL
19	1	CP103041	CASTER AXLE LONG
20	2	TABLE ONE	CASTER SPACER
21	1	CP102579	CASTER PINLOCK CLAMP
22	1	CP102581	CASTER PINLOCK PLATE, STEEL
23	1	CP102580	CASTER PINLOCK MOUNT B
24	1	CP102582	CASTER PINLOCK RELEASE HANDLE
25	1	CP102583	CASTER PINLOCK PIN
26	1	CP102584	SPACER ALUMINUM .500ID X .750OD X .125THICK
27 ^{ΔB}	1	HD100426	8-32 X 1/2" FHCS, STAINLESS STEEL
28	1	HD100427	10-32 X 3/4" SHCS, BLACK ZINC
29 ^{ΔV}	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
30	1	HD100425	8-32 X 1/2" SHCS, BLACK ZINC
31	1	HD100429	SPRING, .328 OD. X .278 ID. X .630 LONG

ASSEMBLIES ON NEXT PAGE



Parts Manual

CASTER PINLOCKS FOR BEARING FORK CONTINUED

TiLite™ Aero X, 2GX, Aero R

ITEM	QTY	PART NO.	DESCRIPTION
1,2A,2B,3A,3B,4-18A,19-31		TABLE ONE	CASTER PINLOCK BEARING FORK, 1/2" EXT, PAIR
1,2A,2B,3A,3B,4-18B,19-31		TABLE ONE	CASTER PINLOCK BEARING FORK SHORT, 1/2" EXT, PAIR
1,2A,2B,3A,3B,4-18C,19-31		TABLE TWO	CASTER PINLOCK BEARING FORK TALL, 1/2" EXT, PAIR

TABLE ONE - SPACERS AND STD/SHORT ASSEMBLIES


	INSIDE SPACER	OUTSIDE SPACER	STD FORK	SHORT FORK
3" MICRO	CP103945	CP103945	AS103608	AS103610
3" VOLCANIC	CP103945	CP103945	AS103608	AS103610
4" PLASTIC W/ POLY	CP103945	CP103945	AS103608	AS103610
4" VOLCANIC	CP103945	CP103945	AS103608	AS103610
4" LITESPEED W/ POLY	CP104405	CP104406	AS111421	AS111422
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111421	AS111422
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111421	AS111422
5" PLASTIC W/ POLY	CP103945	CP103945	AS103608	N/A
5" LITESPEED W/ POLY	CP104405	CP104406	AS111421	N/A
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111421	N/A
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111421	N/A
6" PLASTIC W/ POLY	CP103945	CP103945	AS103608	N/A
6" PLASTIC W/ PNEUMATIC	CP103945	CP103945	AS103608	N/A
6" LITESPEED W/ POLY	CP104405	CP104406	AS111421	N/A
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104405	CP104406	AS111421	N/A
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104405	CP104406	AS111421	N/A
6" LITESPEED W/ PNEUMATIC	CP103945	CP103945	AS103608	N/A

TABLE TWO - SPACERS AND TALL FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	TALL FORK
3" MICRO	CP103007	CP103007	AS103611
3" VOLCANIC	CP103007	CP103007	AS103611
4" PLASTIC W/ POLY	CP103006	CP103006	AS103612
4" VOLCANIC	CP103007	CP103007	AS103611
4" LITESPEED W/ POLY	CP104403	CP104404	AS111423
4" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111423
4" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS111423
5" PLASTIC W/ POLY	CP103006	CP103006	AS111423
5" LITESPEED W/ POLY	CP104403	CP104404	AS111423
5" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111423
5" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS111423
6" PLASTIC W/ POLY	CP103006	CP103006	AS103612
6" PLASTIC W/ PNEUMATIC	CP103006	CP103006	AS103612
6" LITESPEED W/ POLY	CP104403	CP104404	AS111423
6" x 1.5" LITESPEED W/ SOFT ROLL	CP104403	CP104404	AS111423
6" x 1.5" LITESPEED PLASTIC W/ SOFT ROLL	CP104403	CP104404	AS111423
6" LITESPEED W/ PNEUMATIC	CP103006	CP103006	AS103612

CASTER PIN LOCK ASSEMBLY


TiLite™ ZRA

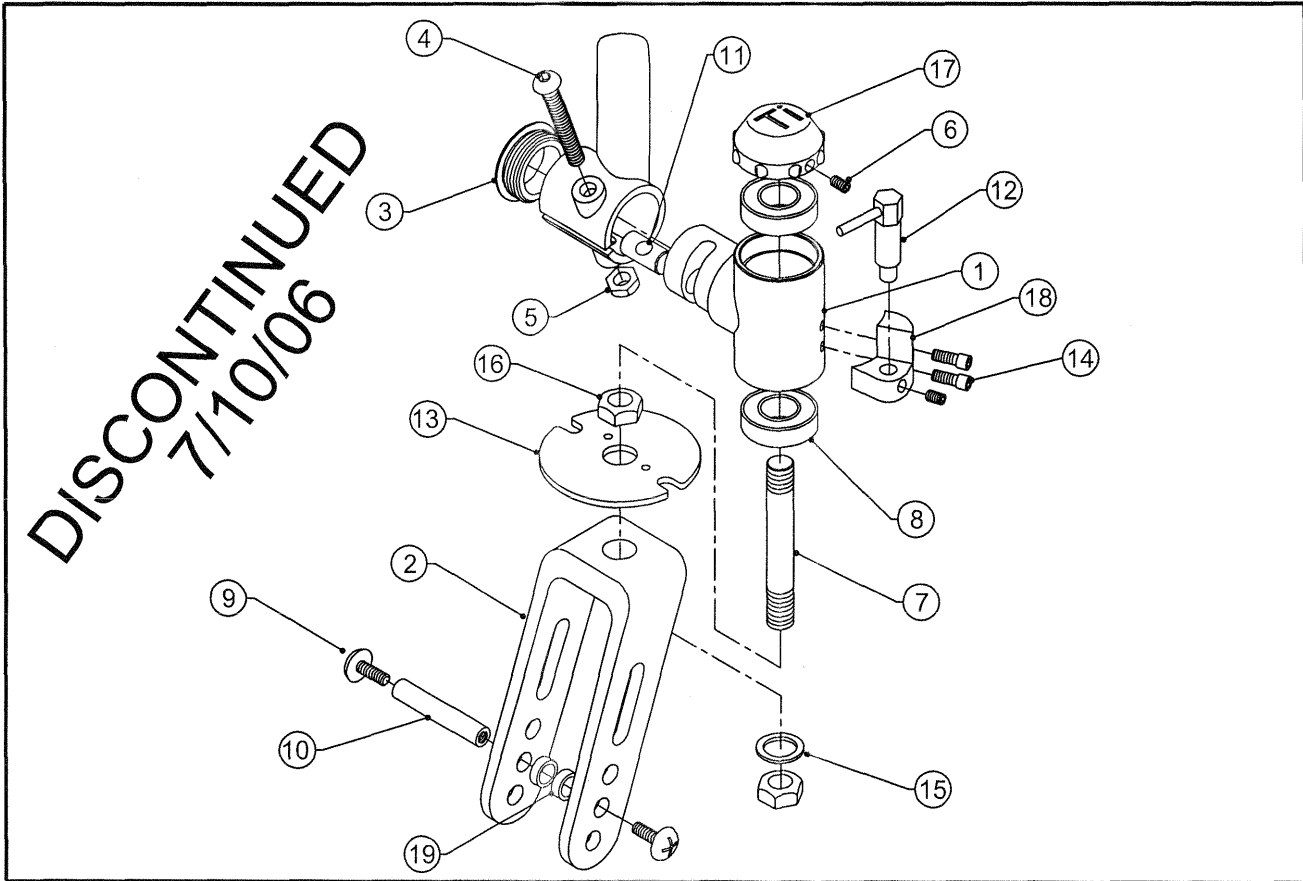

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP102137	CASTER BARREL ZRA FOR CASTER PIN LOCKS RIGHT
1B	1	CP102138	CASTER BARREL ZRA FOR CASTER PIN LOCKS LEFT
2	1	CP100335	5" AND 6" FORK
3	1	CP102064	ENDCAP 1.25 OD X .125 WALL
4	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
5	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
6	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
7	1	CP101413	CASTER STEM STD W/ 1/2" THREAD
8	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
9	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
10	1	HD100085	5"-8" CASTER AXLE
11	1	CP102142	CASTER MOUNT DOWEL Ø.437
12	1	HD100139	L-HANDLE LOCK, 3/8-16 UNC 2A STEEL, CLEAR ZINC PLATE
13	1	CP100685	LOCK RING, STAINLESS STEEL
14	2	HD100136	10-32 X 3/8" SOCKET HEAD CAP SCREW, CLEAR ZINC PLATED
15	2	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06 THK, YELLOW ZINC
16	2	HD100087	1/2-20 UNF LOCK NUT THIN
17	1	CP100928	CASTER CAP (MOLDED)
18A	1	CP100684	CASTER PIN MOUNT, RIGHT
18B	1	CP100687	CASTER PIN MOUNT, LEFT
19A	2	HD100084	438 X .220 SPACER FOR 5" CASTER
19B	2	CP100007	FORK SPACER
1A-17,18A,19A		AS102470	ZRA CASTER PINLOCK FORK ASSEMBLY RIGHT FOR 5" & 6" CASTERS
1B-17,18B,19A		AS102471	ZRA CASTER PINLOCK FORK ASSEMBLY LEFT FOR 5" & 6" CASTERS
1A-17,18A,19B		AS102472	ZRA CASTER PINLOCK FORK ASSEMBLY RIGHT FOR 3" & 4" CASTERS
1B-17,18B,19B		AS102473	ZRA CASTER PINLOCK FORK ASSEMBLY LEFT FOR 3" & 4" CASTERS

FROG LEG® FORK ASSEMBLY


TiLite™ ZR, TR(SERIES 2), YR(SERIES 2), TX

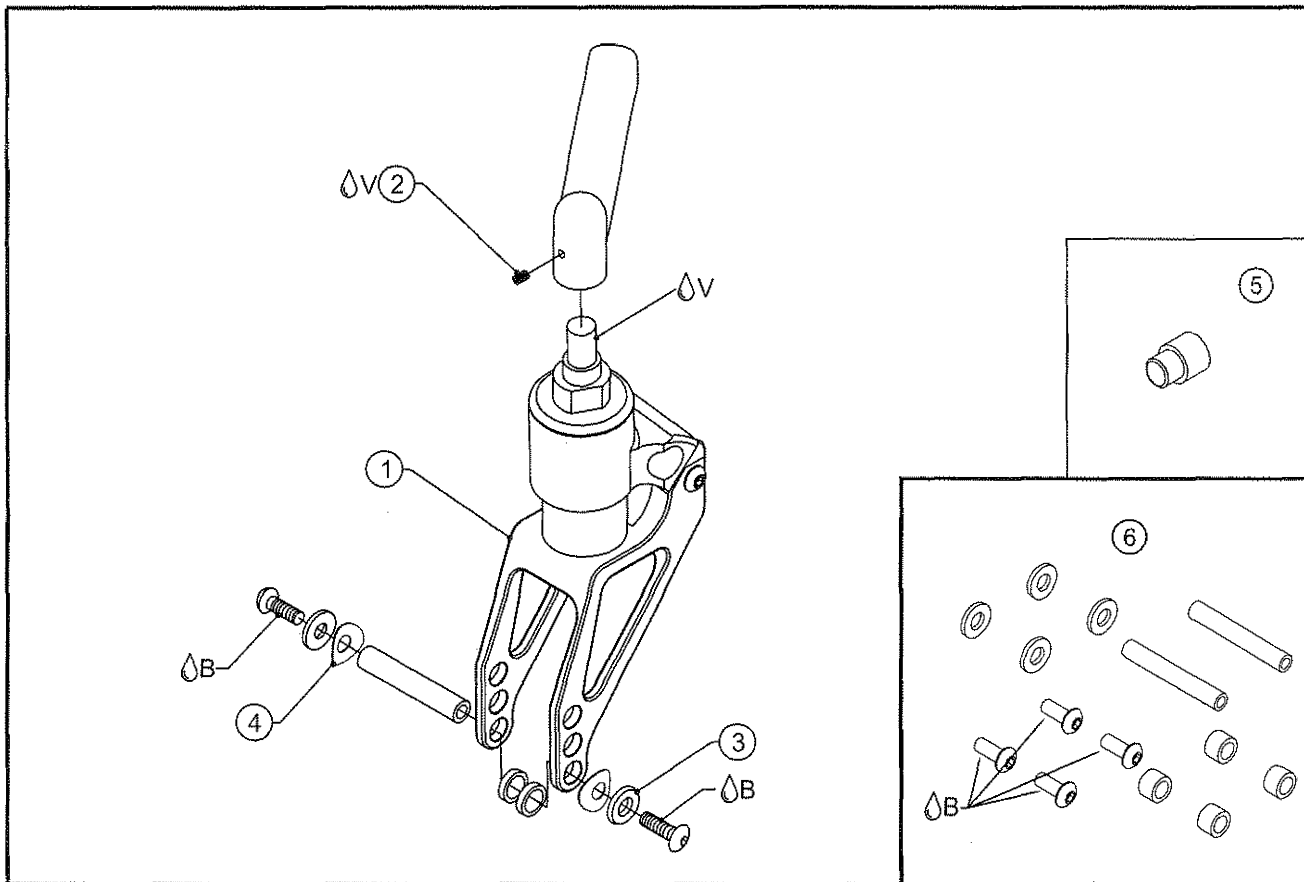

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP102143	FIXED FROGLEG FORK
2V	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
3	2	HD100098	5/16" AN WASHER .312 ID X .562OD X .06 THICK,CLEAR ZINC PLATED
4	2	HD100645	.562 OD X ..250 ID X .020 THK, C SPRING WASHER
5A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
5B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
6	1	CP103522	FROG LEGS HARDWARE PACK
1-4		AS102817	FIXED FROG LEG FORK ASSEMBLY
1-4,5A,5B		AS111433	FIXED FROG LEG FORK ASSEMBLY (PERFORMANCE CASTER)

FROG LEG® FORK ASSEMBLY


TiLite™ ZRA

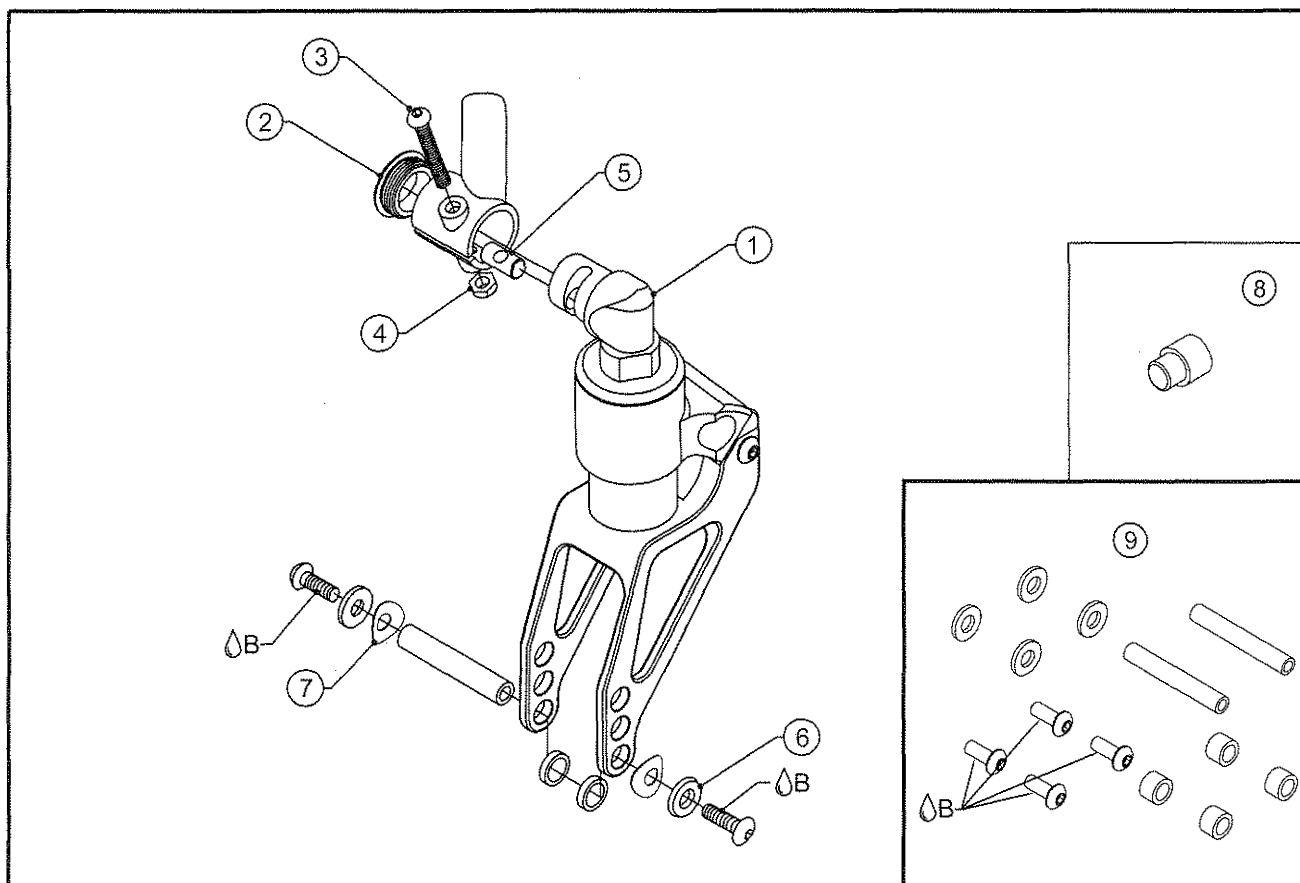

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS102858	ZRA FROG LEGS FORK SUB-ASSEMBLY LEFT
1B	1	AS102859	ZRA FROG LEGS FORK SUB-ASSEMBLY RIGHT
2	1	CP102064	ENDCAP 1.25 OD X .125 WALL
3	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
4	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
5	1	CP102142	CASTER MOUNT DOWEL Ø.437
6	2	HD100098	5/16" AN WASHER .312 ID X .562OD X .06 THICK,CLEAR ZINC PLATED
7	2	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
8A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
8B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
9	1	CP103522	FROG LEGS HARDWARE PACK
1A, 2-7		AS102612	ZRA FROG LEG FORK ASSEMBLY LEFT FOR 3"-6" CASTER
1B, 2-7		AS102613	ZRA FROG LEG FORK ASSEMBLY RIGHT FOR 3"-6" CASTER
1A, 2-7,8A,8B		AS111434	ZRA FROG LEG FORK ASSY LEFT FOR 3"-6" CASTER (PERFORMANCE CASTER)
1B, 2-7,8A,8B		AS111435	ZRA FROG LEG FORK ASSY RIGHT FOR 3"-6" CASTER (PERFORMANCE CASTER)

FROGLEG FORK ASSEMBLY


TiLite™ ZRA (SERIES 2)

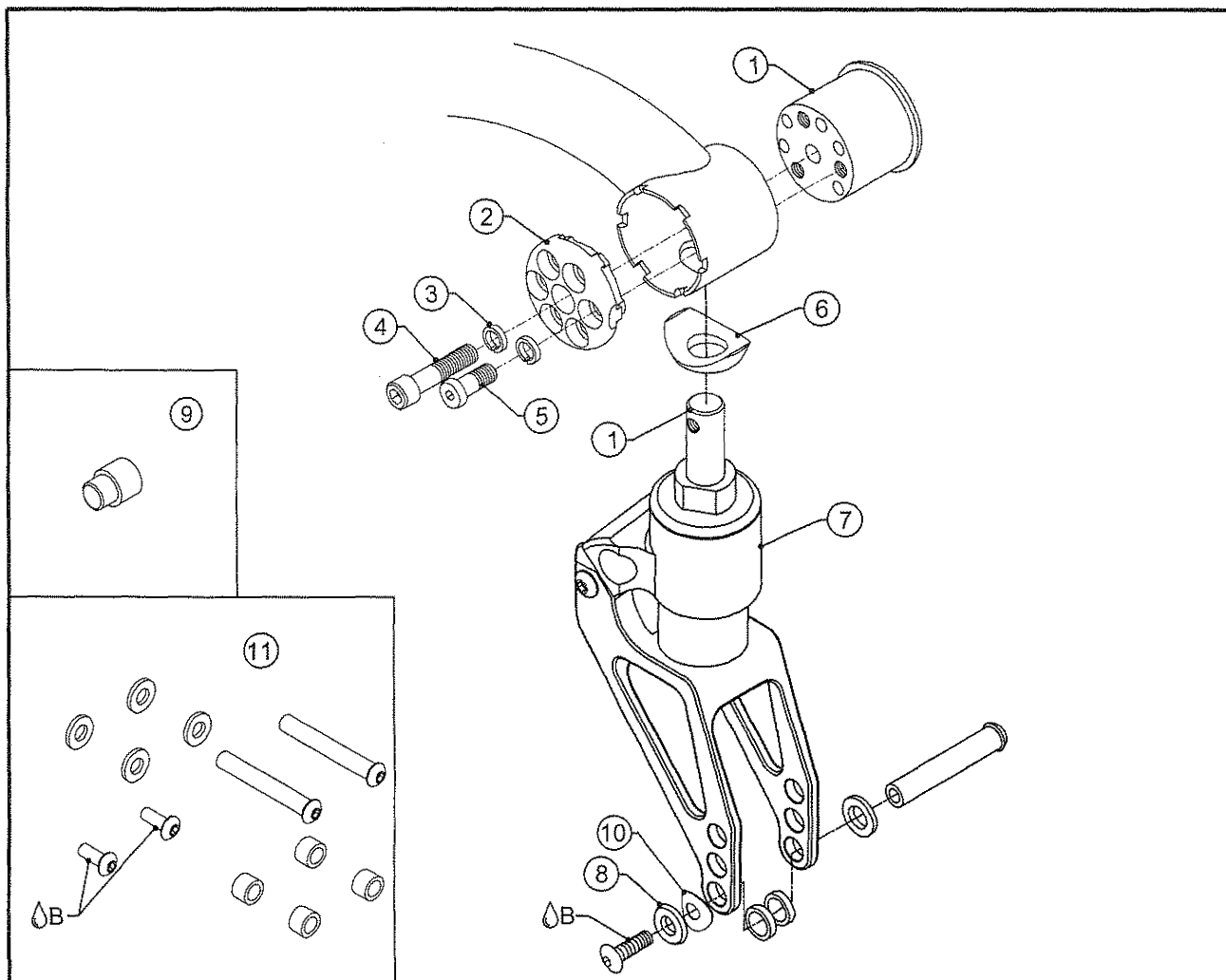

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP104298	CASTER MOUNT, RIGHT
1B	1	CP104299	CASTER MOUNT, LEFT
2A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
2B	1	CP104295	Z2 CASTER ENDCAP, LEFT
3	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
4	1	HD100677	1/4-28 X 1-3/16" W/ 3/4" THREAD SHCS, BLACK ZINC
5	1	HD100601	1/4-28 X 5/8" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
6	1	CP104568	FROG LEGS CASTER CAP
7	1	CP104597	FROGLEG FORK
8	1	HD100098	5/16" AN WASHER .312 ID X .562OD X .06 THICK,CLEAR ZINC PLATED
9A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
9B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
10	1	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
11	1	CP103522	FROG LEGS HARDWARE PACK
1A,2A,3-8,10		AS110863	FROG LEG FORK ASSY, RIGHT
1B,2B,3-8,10		AS110864	FROG LEG FORK ASSY, LEFT
1A,1B,2A,2B,3-8,10		AS110865	FROG LEG FORK ASSY, PAIR
1A,2A,3-8,9A,9B,10		AS110866	FROG LEG FORK ASSY W/ STEPPED SPACER, RIGHT (PERFORMANCE CASTER)
1B,2B,3-8,9A,9B,10		AS110867	FROG LEG FORK ASSY W/ STEPPED SPACER, LEFT (PERFORMANCE CASTER)
1A,1B,2A,2B,3-8,9A,9B,10		AS110868	FROG LEG FORK ASSY W/ STEPPED SPACER, PAIR (PERFORMANCE CASTER)

FROG LEG® FORK ASSEMBLY


TiLite™ AERO Z

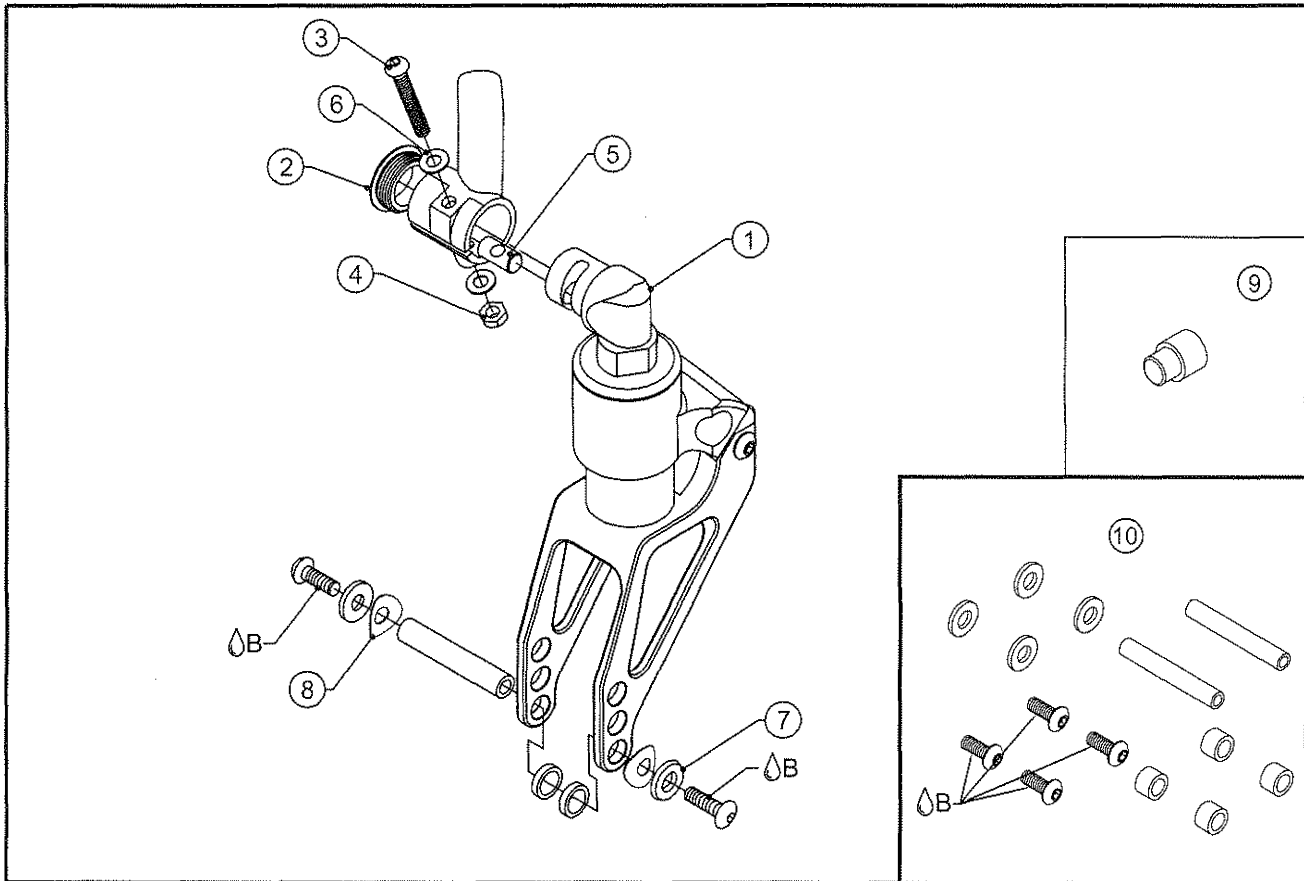

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS102858	FROG LEGS FORK SUB-ASSEMBLY LEFT
1B	1	AS102859	FROG LEGS FORK SUB-ASSEMBLY RIGHT
2	1	CP102064	ENDCAP 1.25 OD X .125 WALL
3	1	HD100324	1/4-28 X 1 3/4 BUTTON HEAD CAP SCREW FULL THREAD
4	1	HD100333	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
5	1	CP102142	CASTER MOUNT DOWEL Ø.437
6	2	HD100402	WASHER 1/4" LIGHT AN (.265 ID X .500 OD X .032 THK)
7	2	HD100098	5/16" AN WASHER .312 ID X .562OD X .06 THICK,CLEAR ZINC PLATED
8	2	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
9A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
9B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
10	1	CP103522	FROG LEGS HARDWARE PACK
1A, 2-8		AS110984	AERO Z FROG LEG FORK ASSEMBLY LEFT
1B, 2-8		AS110985	AERO Z FROG LEG FORK ASSEMBLY RIGHT
1A, 2-8,9A,9B		AS111436	AERO Z FROG LEG FORK ASSEMBLY LEFT (PERFORMANCE CASTER)
1B, 2-8,9A,9B		AS111437	AERO Z FROG LEG FORK ASSEMBLY RIGHT (PERFORMANCE CASTER)

FROGLEG FORK ASSEMBLY


TiLite™ AERO Z (SERIES 2), TWIST, AERO T

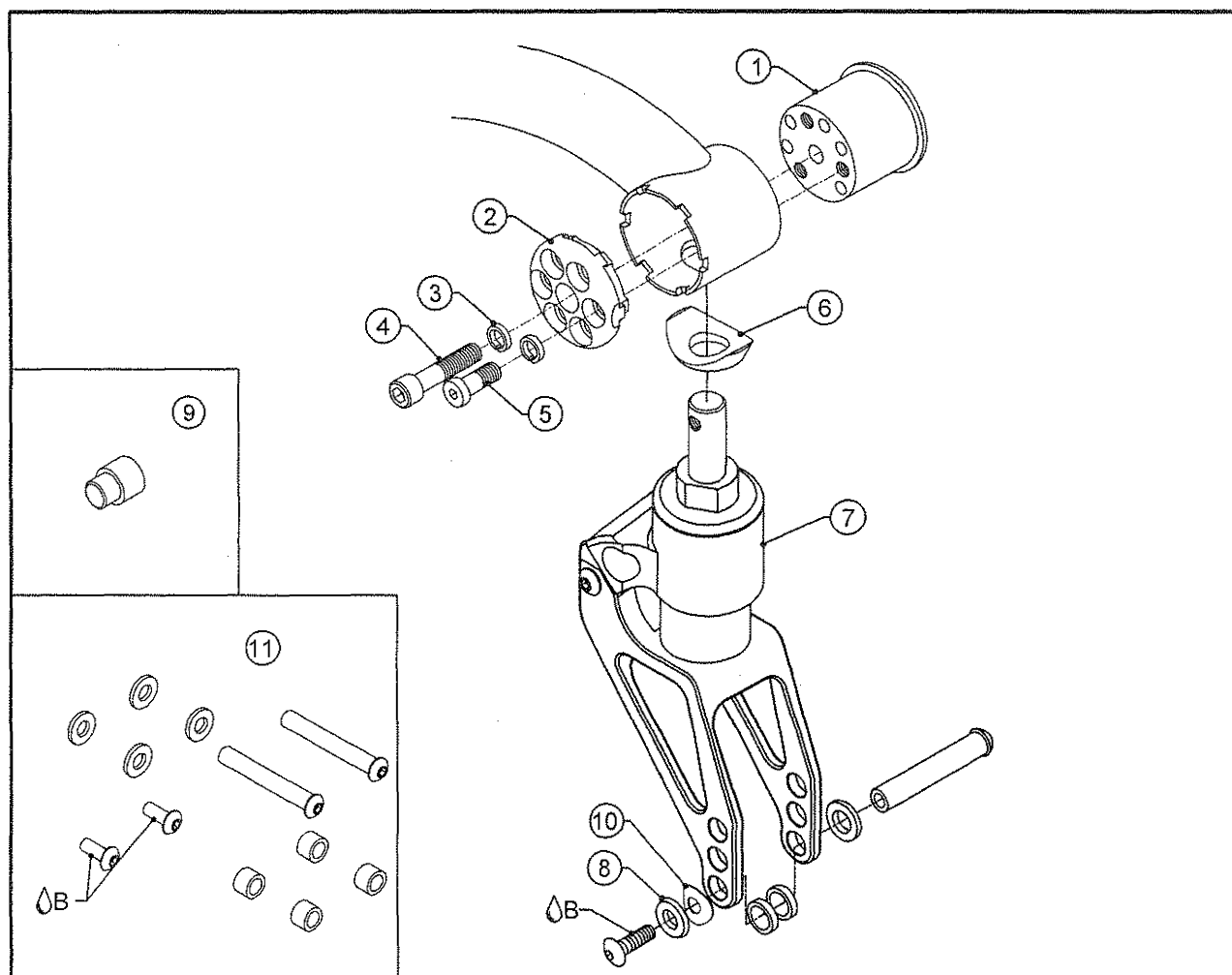

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP104298	CASTER MOUNT, RIGHT
1B	1	CP104299	CASTER MOUNT, LEFT
2A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
2B	1	CP104295	Z2 CASTER ENDCAP, LEFT
3	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
4	1	HD100676	1/4-28 X 1-7/16" W/ 3/4" THREAD SHCS, BLACK ZINC
5	1	HD100661	1/4-28 X 29/32" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
6	1	CP105485	FROG LEGS CASTER CAP
7	1	CP104597	FROGLEG FORK
8	1	HD100098	5/16" AN WASHER .312 ID X .562OD X .06 THICK,CLEAR ZINC PLATED
9A	1	CP104403	CASTER SPACER (NON-DISHED SIDE)
9B	1	CP104404	CASTER SPACER (DISHED SIDE)
10	1	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
11	1	CP103522	FROG LEGS HARDWARE PACK
1A,2A,3-8,10		AS111492	FROG LEG FORK ASSY, RIGHT
1B,2B,3-8,10		AS111493	FROG LEG FORK ASSY, LEFT
1A,1B,2A,2B,3-8,10		AS111494	FROG LEG FORK ASSY, PAIR
1A,2A,3-8,9A,9B,10		AS111495	FROG LEG FORK ASSY W/ STEPPED SPACER, RIGHT
1B,2B,3-8,9A,9B,10		AS111496	FROG LEG FORK ASSY W/ STEPPED SPACER, LEFT
1A,1B,2A,2B,3-8,9A,9B,10		AS111497	FROG LEG FORK ASSY W/ STEPPED SPACER, PAIR

FROG LEGS FORK ASSEMBLY

TiLite™ Aero X, 2GX, Aero R

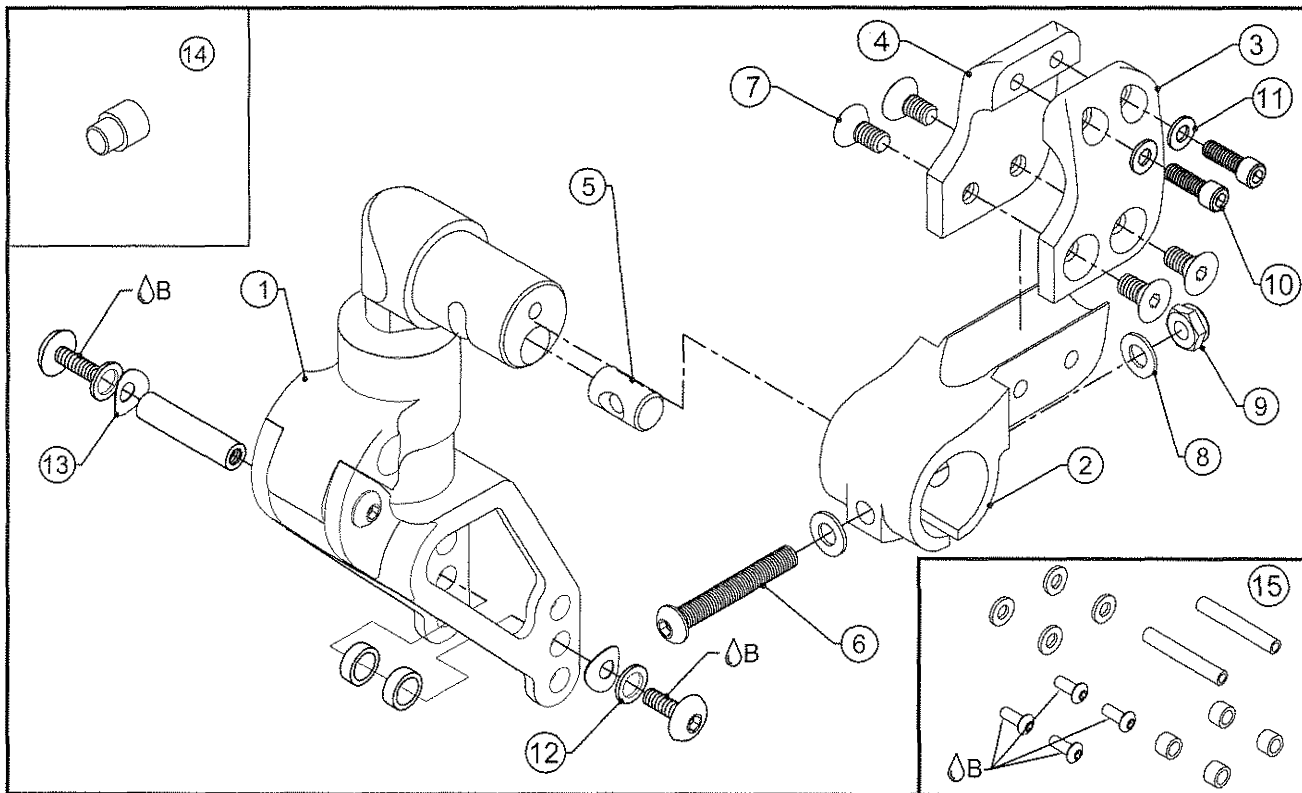


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS103282	FROG LEGS FORK SUB-ASSEMBLY
1B	1	AS103291	FROG LEGS FORK SUB-ASSEMBLY, 1/2" EXT
1C	1	AS103292	FROG LEGS FORK SUB-ASSEMBLY, 1" EXT
2	1	CP103185	CASTER MOUNT
3A	1	CP103895	CASTER MOUNT CLAMP, COUNTERBORED, RIGHT
3B	1	CP103896	CASTER MOUNT CLAMP, COUNTERBORED, LEFT
4A	1	CP103897	CASTER MOUNT CLAMP, TAPPED, RIGHT
4B	1	CP103898	CASTER MOUNT CLAMP, TAPPED, LEFT
5	1	CP102142	CASTER MOUNT DOWEL Ø.437
6	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
7	4	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
8	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
9	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
10	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
11	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
12	2	HD100098	5/16" AN WASHER .312ID .562OD .06 THICK
13	2	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
14A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
14B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
15	1	CP103522	FROG LEGS HARDWARE PACK
1A,2,3A,4A-13		AS103479	FROG LEGS FORK ASSEMBLY, RIGHT
1A,2,3B,4B-13		AS104733	FROG LEGS FORK ASSEMBLY, LEFT
1A,2-13		AS110510	FROG LEGS FORK ASSEMBLY, PAIR
1B,2,3A,4A-13		AS103480	FROG LEGS FORK ASSEMBLY, 1/2" EXT, RIGHT
1B,2,3B,4B-13		AS104734	FROG LEGS FORK ASSEMBLY, 1/2" EXT, LEFT
1B,2-13		AS110511	FROG LEGS FORK ASSEMBLY, 1/2" EXT, PAIR
1C,2,3A,4A-13		AS103481	FROG LEGS FORK ASSEMBLY, 1" EXT, RIGHT
1C,2,3B,4B-13		AS104735	FROG LEGS FORK ASSEMBLY, 1" EXT, LEFT
1C,2-13		AS110512	FROG LEGS FORK ASSEMBLY, 1" EXT, PAIR
1A,2,3A,4A-13,14A,14B		AS111438	FROG LEGS FORK ASSEMBLY, RIGHT (PERFORMANCE CASTER)
1A,2,3B,4B-13,14A,14B		AS111439	FROG LEGS FORK ASSEMBLY, LEFT (PERFORMANCE CASTER)
1A,2-13,14A,14B		AS111440	FROG LEGS FORK ASSEMBLY, PAIR (PERFORMANCE CASTER)
1B,2,3A,4A-13,14A,14B		AS111441	FROG LEGS FORK ASSY, 1/2" EXT, RIGHT (PERFORMANCE CASTER)
1B,2,3B,4B-13,14A,14B		AS111442	FROG LEGS FORK ASSY, 1/2" EXT, LEFT (PERFORMANCE CASTER)
1B,2-13,14A,14B		AS111443	FROG LEGS FORK ASSY, 1/2" EXT, PAIR (PERFORMANCE CASTER)
1C,2,3A,4A-13,14A,14B		AS111444	FROG LEGS FORK ASSY, 1" EXT, RIGHT (PERFORMANCE CASTER)
1C,2,3B,4B-13,14A,14B		AS111445	FROG LEGS FORK ASSY, 1" EXT, LEFT (PERFORMANCE CASTER)
1C,2-13,14A,14B		AS111446	FROG LEGS FORK ASSY, 1" EXT, PAIR (PERFORMANCE CASTER)



FROG LEG FORK ASSEMBLY


HEMI HEIGHT TiLite™ Aero X, 2GX

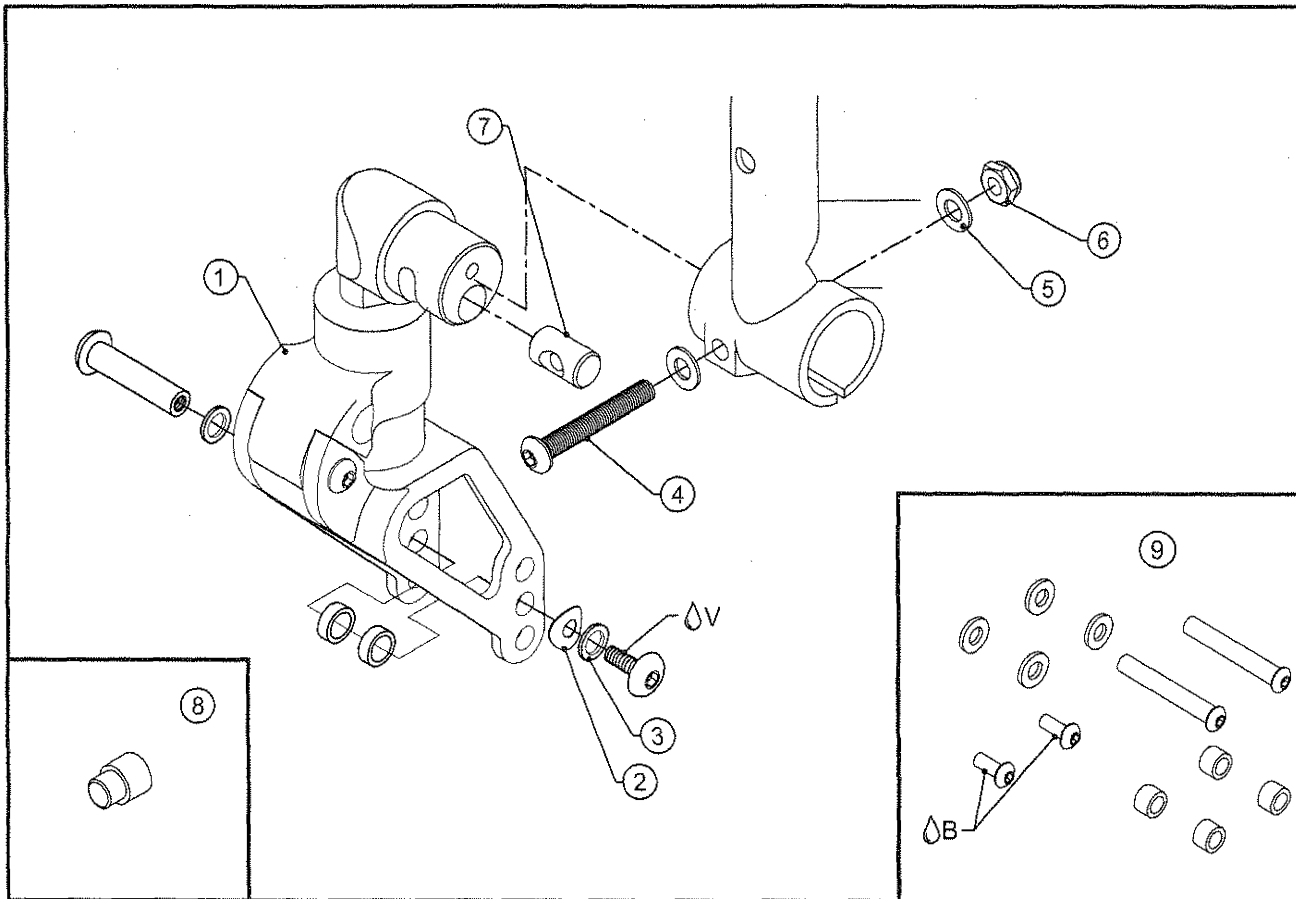

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	AS111997	HEMI FROG LEGS CASTER SUB-ASSEMBLY
1B	1	AS111998	HEMI FROG LEGS CASTER SUB-ASSEMBLY, 1/2" EXT
1C	1	AS111999	HEMI FROG LEGS CASTER SUB-ASSEMBLY, 1" EXT
2	1	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
3	2	HD100098	5/16" AN WASHER .312ID .562OD .06 THICK
4	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
5	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
6	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
7	1	CP102142	CASTER MOUNT DOWEL Ø.437
8A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
8B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
9	1	CP103522	FROG LEGS HARDWARE PACK
1A,2-7		AS112000	HEMI HEIGHT FROG LEG FORK CASTER MOUNT
1B,2-7		AS112001	HEMI HEIGHT FROG LEG FORK CASTER MOUNT W/ 1/2" EXT
1C,2-7		AS112002	HEMI HEIGHT FROG LEG FORK CASTER MOUNT W/ 1" EXT
1A,2-7,8A,8B		AS112003	HEMI HEIGHT FROG LEG FORK CASTER MOUNT(PERFORMANCE CASTER)
1B,2-7,8A,8B		AS112004	HEMI HEIGHT FROG LEG FORK CASTER MOUNT W/ 1/2" EXT(PERFORMANCE CASTER)
1C,2-7,8A,8B		AS112005	HEMI HEIGHT FROG LEG FORK CASTER MOUNT W/ 1" EXT(PERFORMANCE CASTER)

FORK ASSY, FROG LEGS

TiLite™ X, SX, XC, YG, YGS, TR,
TRC, YR, TRA, YRA, EVO

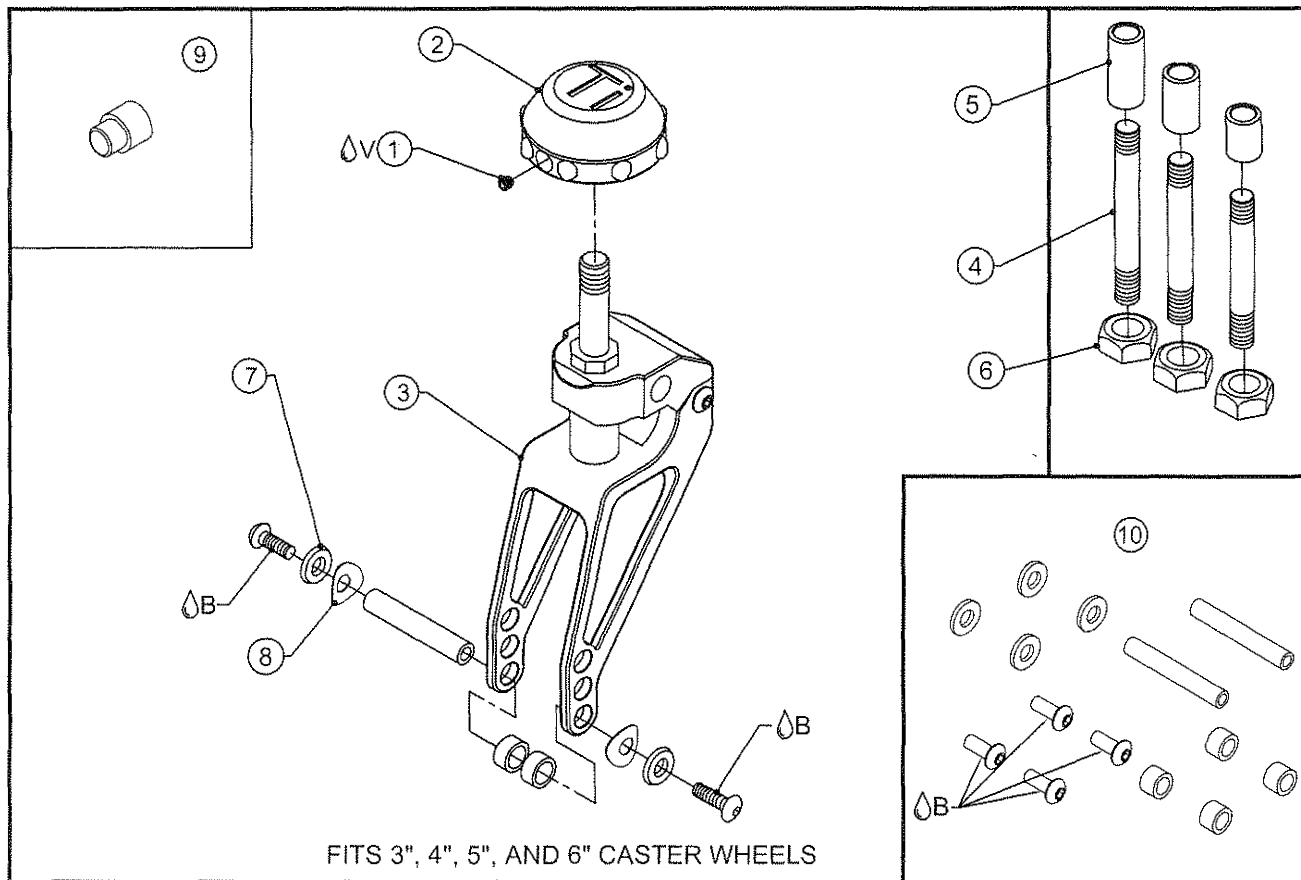


ITEM	QUANTITY	PART NO.	DESCRIPTION
1 ^{ΔV}	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
2	1	CP100928	CASTER CAP (MOLDED)
3	1	CP100906	FROG LEGS FL6 SILVER FOR 6" WHEELS
4A	1	CP101417	FROG LEGS CASTER STEM W/ 1/2" THREAD 1/2" EXTENSION
4B	1	CP101418	FROG LEGS CASTER STEM W/ 1/2" THREAD 1" EXTENSION
4C	1	CP101419	FROG LEGS CASTER STEM W/ 1/2" THREAD 1-1/2" EXTENSION
5A	1	CP100346	CASTER EXTENSION 1/2"
5B	1	CP100347	CASTER EXTENSION 1"
5C	1	CP100660	CASTER EXTENSION 1-1/2"
6	2	HD100086	1/2-20UNF JAM NUT
7	2	HD100098	5/16" AN WASHER .312ID .562OD .06 THICK
8	2	HD100645	.562 OD X .250 ID X .020 THK, C SPRING WASHER
9A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
9B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
10	1	CP103522	FROG LEGS HARDWARE PACK
1-3,7-8		AS101879	6" FROG LEGS STD ASSEMBLY
1-3,4A,5A,6-8		AS101880	6" FROG LEGS 1/2" EXT ASSEMBLY
1-3,4B,5B,6-8		AS101881	6" FROG LEGS 1" EXT ASSEMBLY
1-3,4C,5C,6-8		AS101882	6" FROG LEGS 1-1/2" EXT ASSEMBLY
1-3,7-8,9A,9B		AS111447	6" FROG LEGS STD ASSEMBLY (PERFORMANCE CASTER)
1-3,4A,5A,6-8,9A,9B		AS111448	6" FROG LEGS 1/2" EXT ASSEMBLY (PERFORMANCE CASTER)
1-3,4B,5B,6-8,9A,9B		AS111449	6" FROG LEGS 1" EXT ASSEMBLY (PERFORMANCE CASTER)
1-3,4C,5C,6-8,9A,9B		AS111450	6" FROG LEGS 1-1/2" EXT ASSEMBLY (PERFORMANCE CASTER)



Parts Manual

FORK ASSY, FROG LEGS (DISCONT. 7-12-2004)


ALL CHAIRS

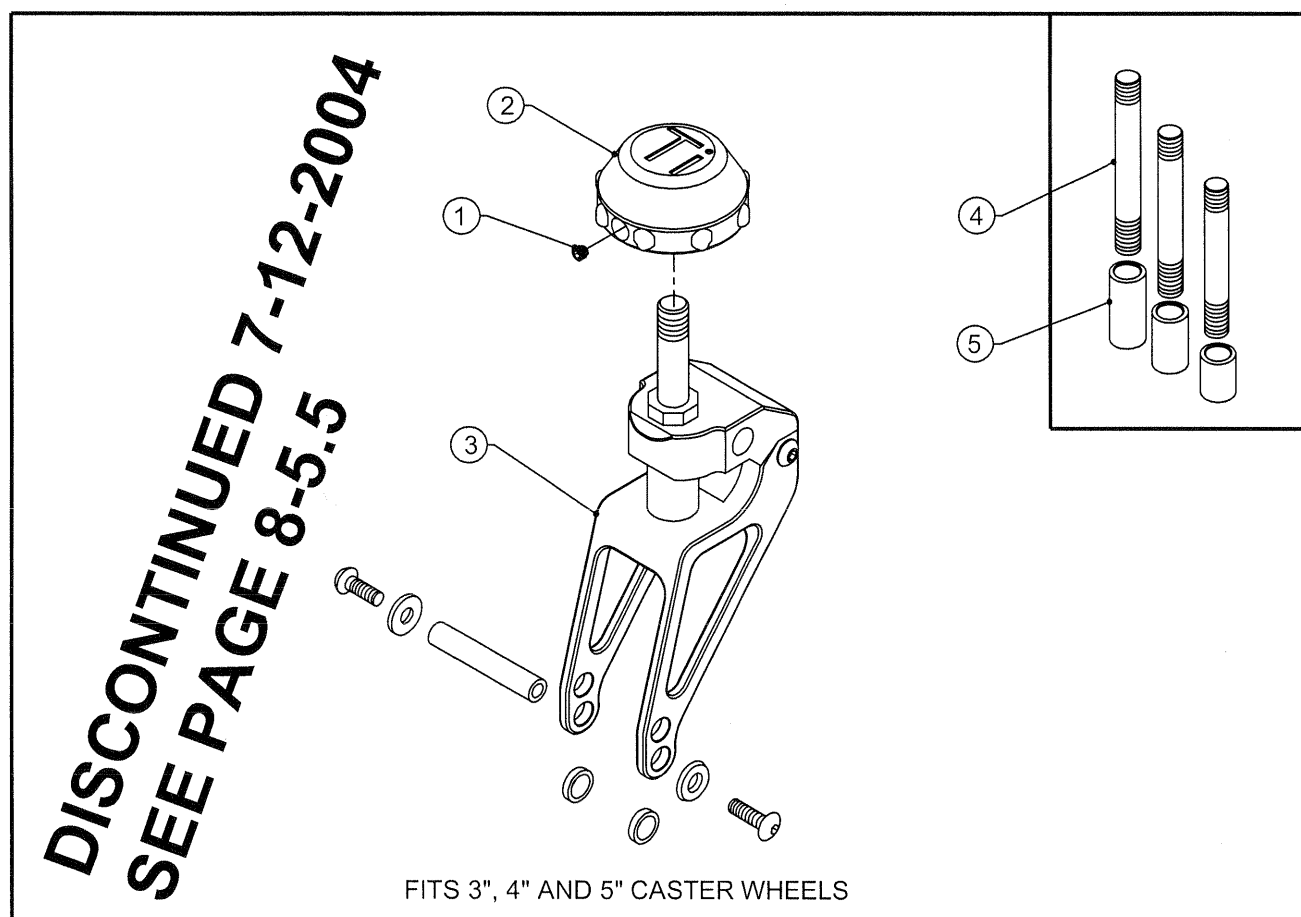

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	HD100132	8-32 X 3/16" SOCKET SET SCREW, NYLON TIP
2	1	CP100928	CASTER CAP (MOLDED)
3	1	CP100637	FROG LEGS MC5 SILVER FOR 5" WHEELS
4A	1	CP101417	FROG LEGS CASTER STEM W/ 1/2" THREAD 1/2" EXTENSION
4B	1	CP101418	FROG LEGS CASTER STEM W/ 1/2" THREAD 1" EXTENSION
4C	1	CP101419	FROG LEGS CASTER STEM W/ 1/2" THREAD 1-1/2" EXTENSION
5A	1	CP100346	CASTER EXTENSION 1/2"
5B	1	CP100347	CASTER EXTENSION 1"
5C	1	CP100660	CASTER EXTENSION 1-1/2"
1-3		AS101104	5" WHEELS FROG LEGS MC5 SILVER ASSY
1-3, 4A, 5A		AS101105	5" WHEELS FROG LEGS MC5 SILVER 1/2" EXT ASSY
1-3, 4B, 5B		AS101106	5" WHEELS FROG LEGS MC5 SILVER 1" EXT ASSY
1-3, 4C, 5C		AS101107	5" WHEELS FROG LEGS MC5 SILVER 1-1/2" EXT ASSY

GLIDE FORK ASSEMBLY

TiLite™ ZR, TR (SERIES 3), TX

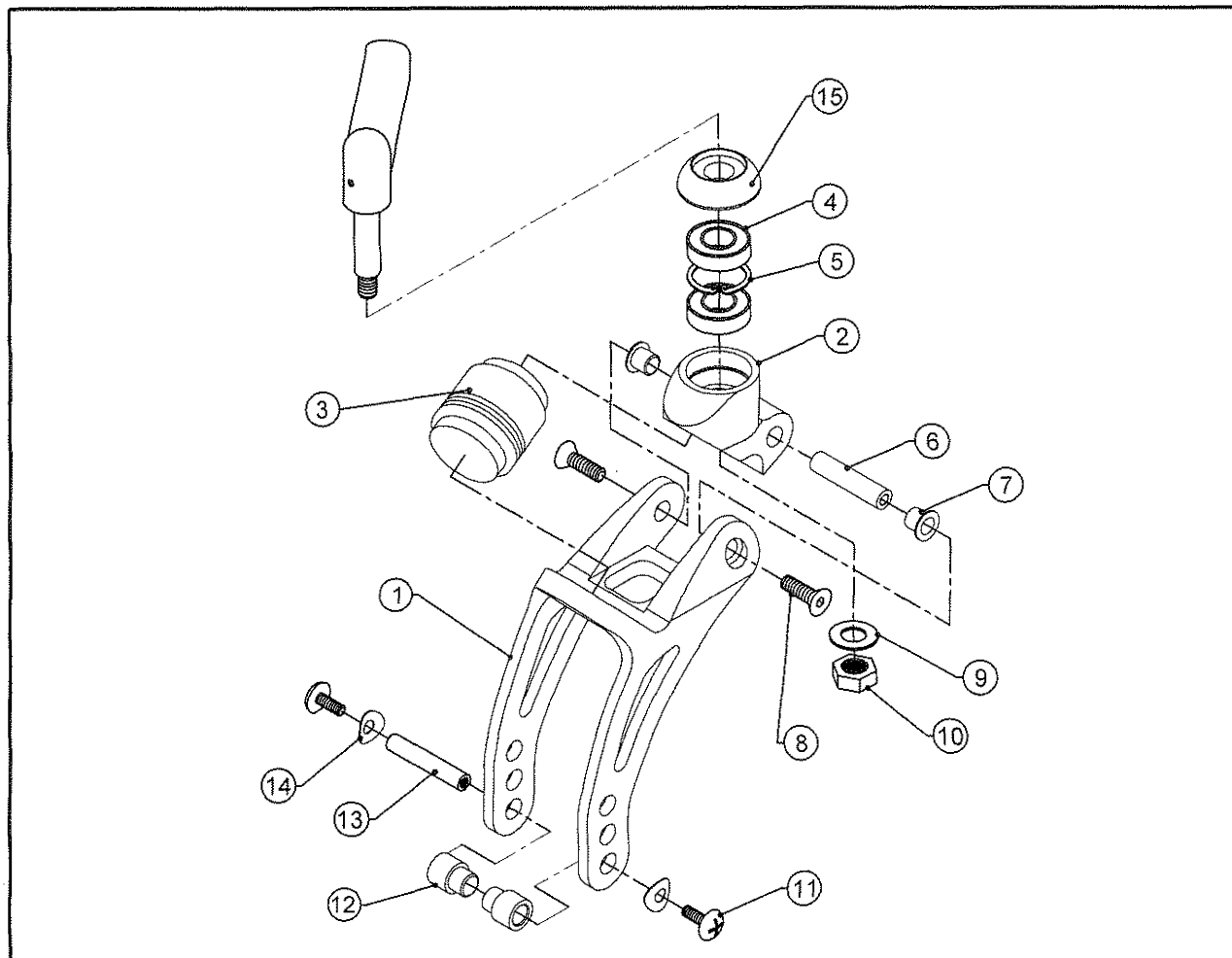

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP107149	GLIDE SUSPENSION FORK BODY
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE, THREAD PATCH
12A	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
12B	1	CP104405	POLY CASTER SPACER (NON-DISHED SIDE)
12C	1	CP104406	POLY CASTER SPACER (DISHED SIDE)
13	1	CP103041	CASTER AXLE LONG
14	2	CP104375	C SPRING WASHER
15	1	CP103744	CASTER CAP, MONO FORK
12A		AS116546	CASTER AXLE SPACERS
12B, 12C		AS116547	POLY CASTER SPACERS
1-3A, 4-12A, 13-14		AS116548	LIGHT AXLE SPACED GLIDE ASSY
1-3B, 4-12A, 13-14		AS116640	MEDIUM AXLE SPACERS GLIDE ASSY
1-3C, 4-12A, 13-14		AS116732	HEAVY AXLE SPACERS GLIDE ASSY
1-3A, 4-12B, 12C-14		AS116594	LIGHT POLY SPACED GLIDE ASSY
1-3B, 4-12B, 12C-14		AS116686	MEDIUM POLY SPACERS GLIDE ASSY
1-3C, 4-12B, 12C-14		AS116778	HEAVY POLY SPACERS GLIDE ASSY



GLIDE FORK ASSEMBLY

TiLite™ ZR, TR (SERIES 3), TX


TABLE ONE - SPACERS

	SPACER ASSEMBLY
3" x .75" MICRO	AS116546
3" x .75" LIGHT-UP MICRO-CASTER	AS116546
3" x .75" LITESPEED BILLET AL WHEEL W/SOFT ROLL	AS116547
4" x .75" PLASTIC WHEEL W/ POLY	AS116546
4" x .75" LIGHT-UP MICRO-CASTER	AS116546
4" x .75" LITESPEED BILLET AL WHEEL W/POLY	AS116547
4" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
4" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
5" x 1" PLASTIC WHEEL W/ POLY	AS116546
5" x 1.5" LITESPEED PLASTIC WHEEL W/ SOFT ROLL	AS116547
5" x .75" LITESPEED BILLET AL WHEEL W/ POLY	AS116547
5" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
6" x 1" PLASTIC WHEEL W/ POLY	AS116546
6" x 1" LITESPEED BILLET AL WHEEL W/POLY	AS116547
6" x 1.25" LITESPEED BILLET AL WHEEL W/PNEUMATIC	AS116547
6" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
6" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547



GLIDE SINGLE FORK ASSEMBLY

TiLite™ ZR, TR (SERIES 3), TX

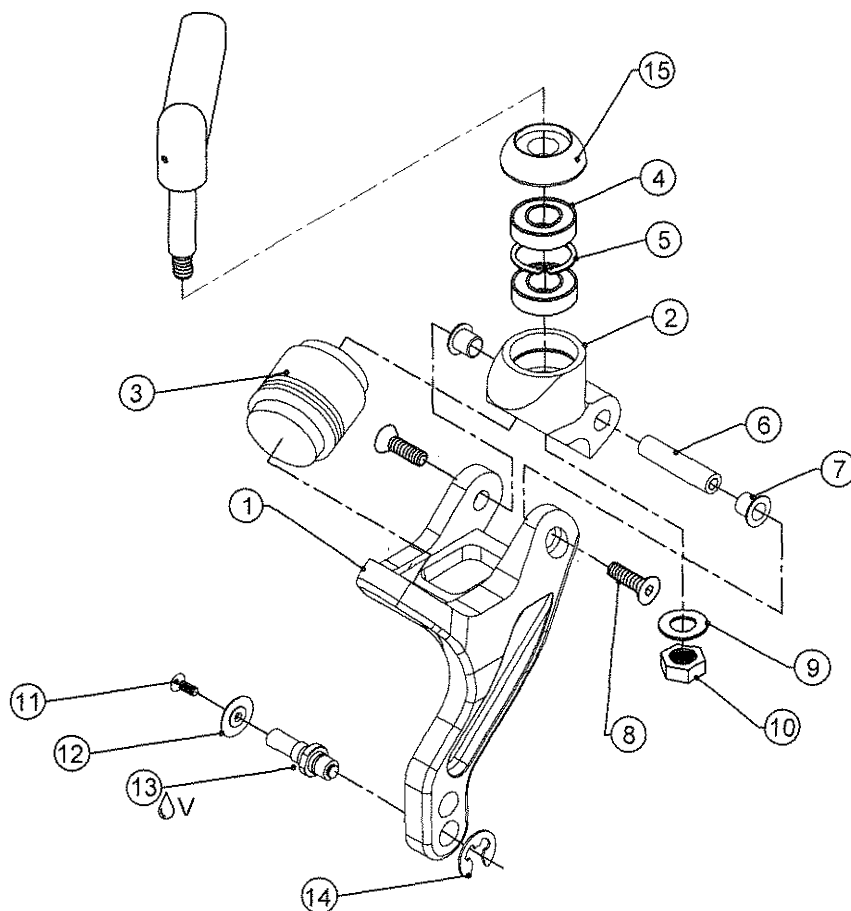


ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP107147	GLIDE SUSPENSION FORK SINGLE SIDED LEFT
1B	1	CP107146	GLIDE SUSPENSION FORK SINGLE SIDED RIGHT
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW WITH THREAD PATCH
11B	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
12A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
12B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
13	1	CP104308	SINGLE FORK AXLE
14	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
15	1	CP103744	CASTER CAP, MONO FORK
1A-3A, 4-11A, 12A-14		AS116100	LIGHT LEFT PRONG CLEAR GLIDE ASSY
1A-3A, 4-11B, 12B-14		AS116159	LIGHT LEFT PRONG BLACK GLIDE ASSY
1B-3A, 4-11A, 12A-14		AS116336	LIGHT RIGHT PRONG CLEAR GLIDE ASSY
1B-3A, 4-11B, 12B-14		AS116371	LIGHT RIGHT PRONG BLACK GLIDE ASSEMBLY
1A-3B, 4-11A, 12A-14		AS116195	MED LEFT PRONG CLEAR GLIDE ASSY
1A-3B, 4-11B, 12B-14		AS116230	MED LEFT PRONG BLACK GLIDE ASSY
1B-3B, 4-11A, 12A-14		AS116406	MED RIGHT PRONG CLEAR GLIDE ASSY
1B-3B, 4-11B, 12B-14		AS116441	MED RIGHT PRONG BLACK GLIDE ASSY
1A-3C, 4-11A, 12A-14		AS116265	HEAVY LEFT PRONG CLEAR GLIDE ASSY
1A-3C, 4-11B, 12B-14		AS116300	HEAVY LEFT PRONG BLACK GLIDE ASSY
1B-3C, 4-11A, 12A-14		AS116476	HEAVY RIGHT PRONG CLEAR GLIDE ASSY
1B-3C, 4-11B, 12B-14		AS116511	HEAVY RIGHT PRONG BLACK GLIDE ASSY



GLIDE FORK ASSEMBLY

TiLite™ ZRA (SERIES 2), TRA (SERIES 2)

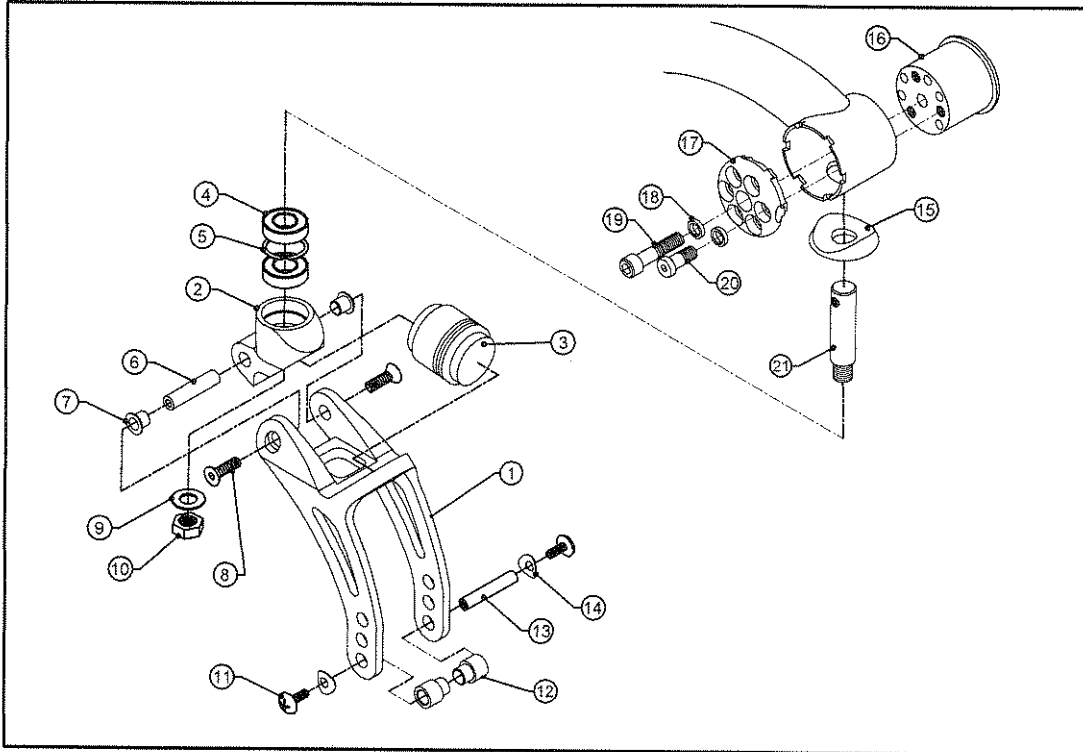

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP107149	GLIDE SUSPENSION FORK BODY
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8" O.D. X 5/16" W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	SUSPENSION FORK BUSHING
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
12A	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
12B	1	CP104405	POLY CASTER SPACER (NON-DISHED SIDE)
12C	1	CP104406	POLY CASTER SPACER (DISHED SIDE)
13	1	CP103041	CASTER AXLE LONG
14	2	CP104375	C SPRING WASHER
15A	1	CP104293	CASTER CAP
15B	1	CP104304	CASTER CAP 1/2" EXT
15C	1	CP104305	CASTER CAP 1" EXT
16A	1	CP104298	CASTER MOUNT, RIGHT
16B	1	CP104299	CASTER MOUNT, LEFT
17A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
17B	1	CP104295	Z2 CASTER ENDCAP, LEFT
18	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
19	1	HD100677	1/4-28 X 1-3/16" W/ 3/4" THREAD SHCS, BLACK ZINC
20	1	HD100601	1/4-28 X 5/8" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
21A	1	CP104292	BEARING FORK STEM
21B	1	CP104302	BEARING FORK STEM W/ 1/2" EXT
21C	1	CP104303	BEARING FORK STEM W/ 1" EXT
21D	1	CP104372	TITANIUM BEARING FORK STEM
12A		AS116546	CASTER AXLE SPACERS
12B, 12C		AS116547	POLY CASTER SPACERS
1-3A, 4-12A, 13-14		AS116548	LIGHT AXLE SPACED GLIDE ASSY
1-3B, 4-12A, 13-14		AS116640	MEDIUM AXLE SPACERS GLIDE ASSY
1-3C, 4-12A, 13-14		AS116732	HEAVY AXLE SPACERS GLIDE ASSY
1-3A, 4-12B, 12C-14		AS116594	LIGHT POLY SPACED GLIDE ASSY
1-3B, 4-12B, 12C-14		AS116686	MEDIUM POLY SPACERS GLIDE ASSY
1-3C, 4-12B, 12C-14		AS116778	HEAVY POLY SPACERS GLIDE ASSY

GLIDE FORK ASSEMBLY


TiLite™ ZRA (SERIES 2), TRA (SERIES 2)

TABLE ONE - SPACERS

	SPACER ASSEMBLY
3" x .75" MICRO	AS116546
3" x .75" LIGHT-UP MICRO-CASTER	AS116546
3" x .75" LITESPEED BILLET AL WHEEL W/SOFT ROLL	AS116547
4" x .75" PLASTIC WHEEL W/ POLY	AS116546
4" x .75" LIGHT-UP MICRO-CASTER	AS116546
4" x .75" LITESPEED BILLET AL WHEEL W/POLY	AS116547
4" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
4" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
5" x 1" PLASTIC WHEEL W/ POLY	AS116546
5" x 1.5" LITESPEED PLASTIC WHEEL W/ SOFT ROLL	AS116547
5" x .75" LITESPEED BILLET AL WHEEL W/ POLY	AS116547
5" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
6" x 1" PLASTIC WHEEL W/ POLY	AS116546
6" x 1" LITESPEED BILLET AL WHEEL W/POLY	AS116547
6" x 1.25" LITESPEED BILLET AL WHEEL W/PNEUMATIC	AS116547
6" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
6" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547

GLIDE SINGLE FORK ASSEMBLY


TiLite™ ZRA (SERIES 2)

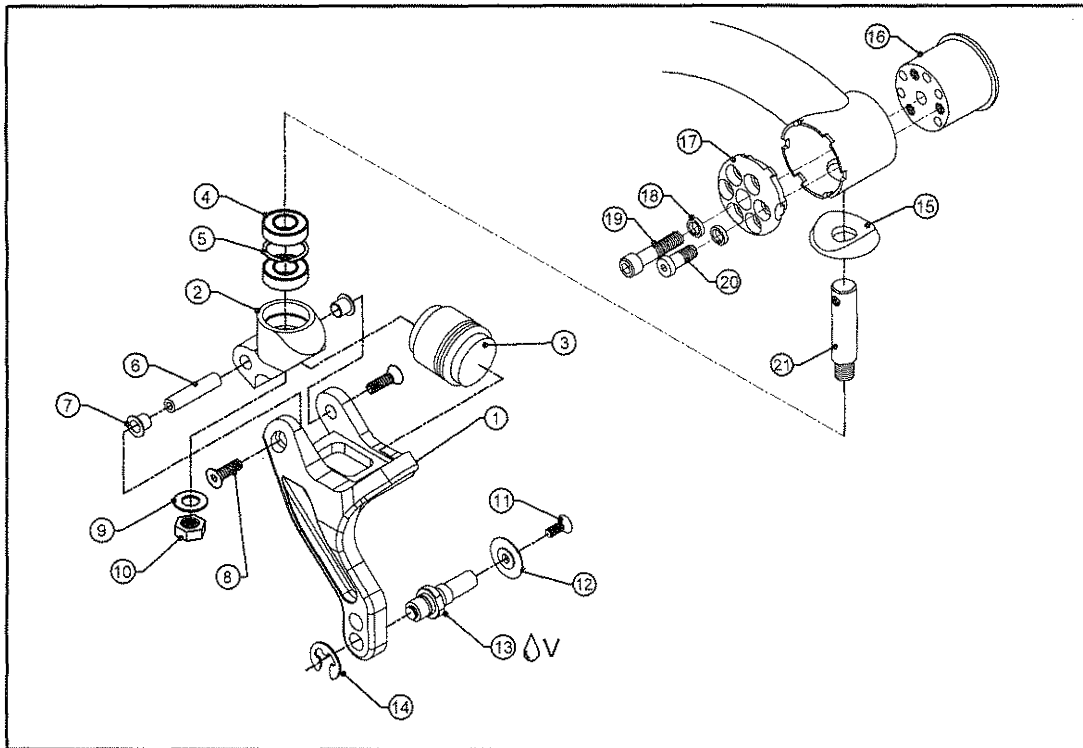

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP107147	GLIDE SUSPENSION FORK SINGLE SIDED LEFT
1B	1	CP107146	GLIDE SUSPENSION FORK SINGLE SIDED RIGHT
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW WITH THREAD PATCH
11B	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
12A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
12B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
13	1	CP104308	SINGLE FORK AXLE
14	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
15A	1	CP104293	CASTER CAP
15B	1	CP104304	CASTER CAP 1/2" EXT
15C	1	CP104305	CASTER CAP 1" EXT
16A	1	CP104298	CASTER MOUNT, RIGHT
16B	1	CP104299	CASTER MOUNT, LEFT
17A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
17B	1	CP104295	Z2 CASTER ENDCAP, LEFT
18	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
19	1	HD100677	1/4-28 X 1-3/16" W/ 3/4" THREAD SHCS, BLACK ZINC
20	1	HD100601	1/4-28 X 5/8" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK ZINC
21A	1	CP104292	BEARING FORK STEM
21B	1	CP104302	BEARING FORK STEM W/ 1/2" EXT
21C	1	CP104303	BEARING FORK STEM W/ 1" EXT
21D	1	CP104372	TITANIUM BEARING FORK STEM
1A-3A, 4-11A, 12A-14		AS116100	LIGHT LEFT PRONG CLEAR GLIDE ASSY
1A-3A, 4-11B, 12B-14		AS116159	LIGHT LEFT PRONG BLACK GLIDE ASSY
1B-3A, 4-11A, 12A-14		AS116336	LIGHT RIGHT PRONG CLEAR GLIDE ASSY
1B-3A, 4-11B, 12B-14		AS116371	LIGHT RIGHT PRONG BLACK GLIDE ASSY
1A-3B, 4-11A, 12A-14		AS116195	MED LEFT PRONG CLEAR GLIDE ASSY
1A-3B, 4-11B, 12B-14		AS116230	MED LEFT PRONG BLACK GLIDE ASSY
1B-3B, 4-11A, 12A-14		AS116406	MED RIGHT PRONG CLEAR GLIDE ASSY
1B-3B, 4-11B, 12B-14		AS116441	MED RIGHT PRONG BLACK GLIDE ASSY
1A-3C, 4-11A, 12A-14		AS116265	HEAVY LEFT PRONG CLEAR GLIDE ASSY
1A-3C, 4-11B, 12B-14		AS116300	HEAVY LEFT PRONG BLACK GLIDE ASSY
1B-3C, 4-11A, 12A-14		AS116476	HEAVY RIGHT PRONG CLEAR GLIDE ASSY
1B-3C, 4-11B, 12B-14		AS116511	HEAVY RIGHT PRONG BLACK GLIDE ASSY

GLIDE FORK ASSEMBLY

TiLite™ AERO Z (SERIES 2), TWIST, AERO T

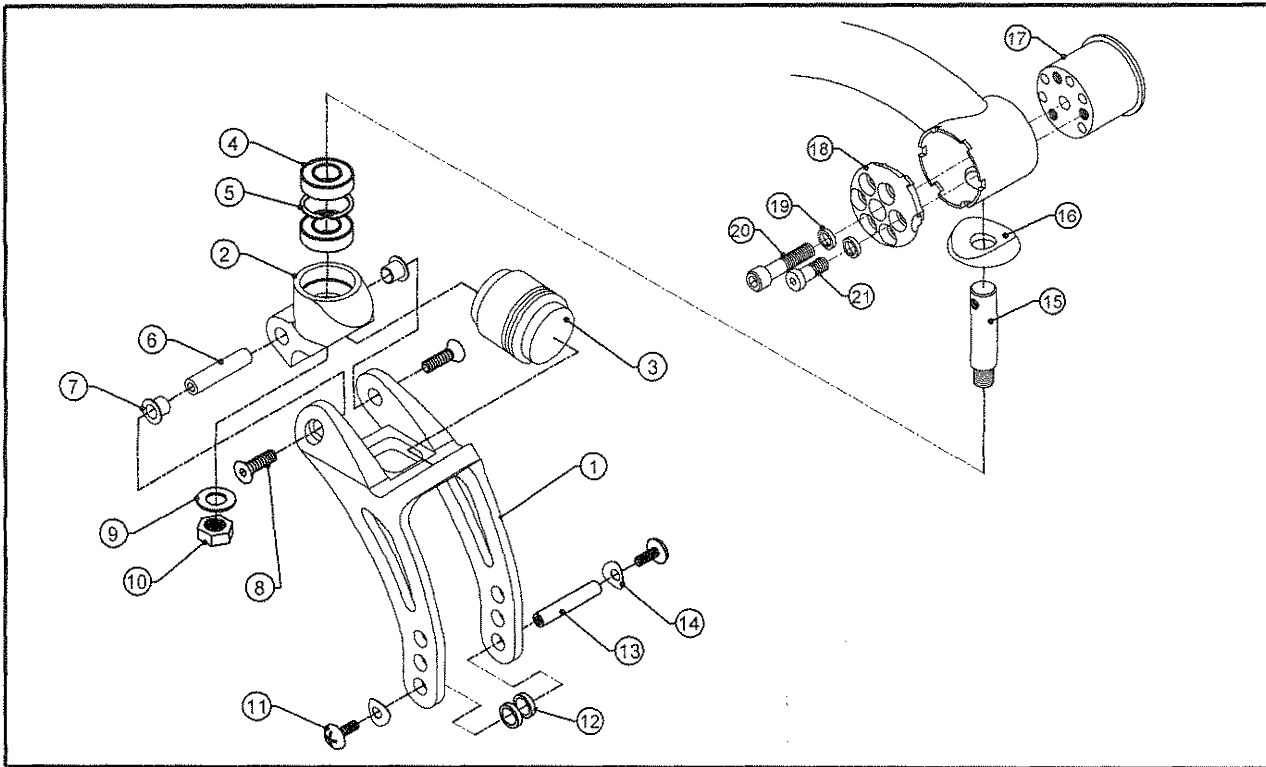

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP107149	GLIDE SUSPENSION FORK BODY
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	SUSPENSION FORK BUSHING
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE, THREAD PATCH
12A	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
12B	1	CP104405	POLY CASTER SPACER (NON-DISHED SIDE)
12C	1	CP104406	POLY CASTER SPACER (DISHED SIDE)
13	1	CP103041	CASTER AXLE LONG
14	2	CP104375	C SPRING WASHER
15A	1	CP104292	BEARING FORK STEM
15B	1	CP104302	BEARING FORK STEM W/ 1/2" EXT
15C	1	CP104303	BEARING FORK STEM W/ 1" EXT
15D	1	CP104372	TITANIUM BEARING FORK STEM
16A	1	CP105482	CASTER CAP
16B	1	CP105483	CASTER CAP 1/2" EXT
16C	1	CP105484	CASTER CAP 1" EXT
17A	1	CP104298	CASTER MOUNT, RIGHT
17B	1	CP104299	CASTER MOUNT, LEFT
18A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
18B	1	CP104295	Z2 CASTER ENDCAP, LEFT
19	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
20	1	HD100676	1/4-28 X 1-7/16" W/ 3/4" THREAD SHCS, BLACK ZINC
21	1	HD100661	1/4-28 X 29/32" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
12A		AS116546	CASTER AXLE SPACERS
12B, 12C		AS116547	POLY CASTER SPACERS
1-3A, 4-12A, 13-14		AS116548	LIGHT AXLE SPACED GLIDE ASSY
1-3B, 4-12A, 13-14		AS116640	MEDIUM AXLE SPACERS GLIDE ASSY
1-3C, 4-12A, 13-14		AS116732	HEAVY AXLE SPACERS GLIDE ASSY
1-3A, 4-12B, 12C-14		AS116594	LIGHT POLY SPACED GLIDE ASSY
1-3B, 4-12B, 12C-14		AS116686	MEDIUM POLY SPACERS GLIDE ASSY
1-3C, 4-12B, 12C-14		AS116778	HEAVY POLY SPACERS GLIDE ASSY



GLIDE FORK ASSEMBLY


TiLite™ AERO Z (SERIES 2), TWIST, AERO T

TABLE ONE - SPACERS

	SPACER ASSEMBLY
3" x .75" MICRO	AS116546
3" x .75" LIGHT-UP MICRO-CASTER	AS116546
3" x .75" LITESPEED BILLET AL WHEEL W/SOFT ROLL	AS116547
4" x .75" PLASTIC WHEEL W/ POLY	AS116546
4" x .75" LIGHT-UP MICRO-CASTER	AS116546
4" x .75" LITESPEED BILLET AL WHEEL W/POLY	AS116547
4" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
4" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
5" x 1" PLASTIC WHEEL W/ POLY	AS116546
5" x 1.5" LITESPEED PLASTIC WHEEL W/ SOFT ROLL	AS116547
5" x .75" LITESPEED BILLET AL WHEEL W/ POLY	AS116547
5" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
6" x 1" PLASTIC WHEEL W/ POLY	AS116546
6" x 1" LITESPEED BILLET AL WHEEL W/POLY	AS116547
6" x 1.25" LITESPEED BILLET AL WHEEL W/PNEUMATIC	AS116547
6" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
6" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547

GLIDE SINGLE FORK ASSEMBLY


TiLite™ AERO Z (SERIES 2), TWIST, AERO T

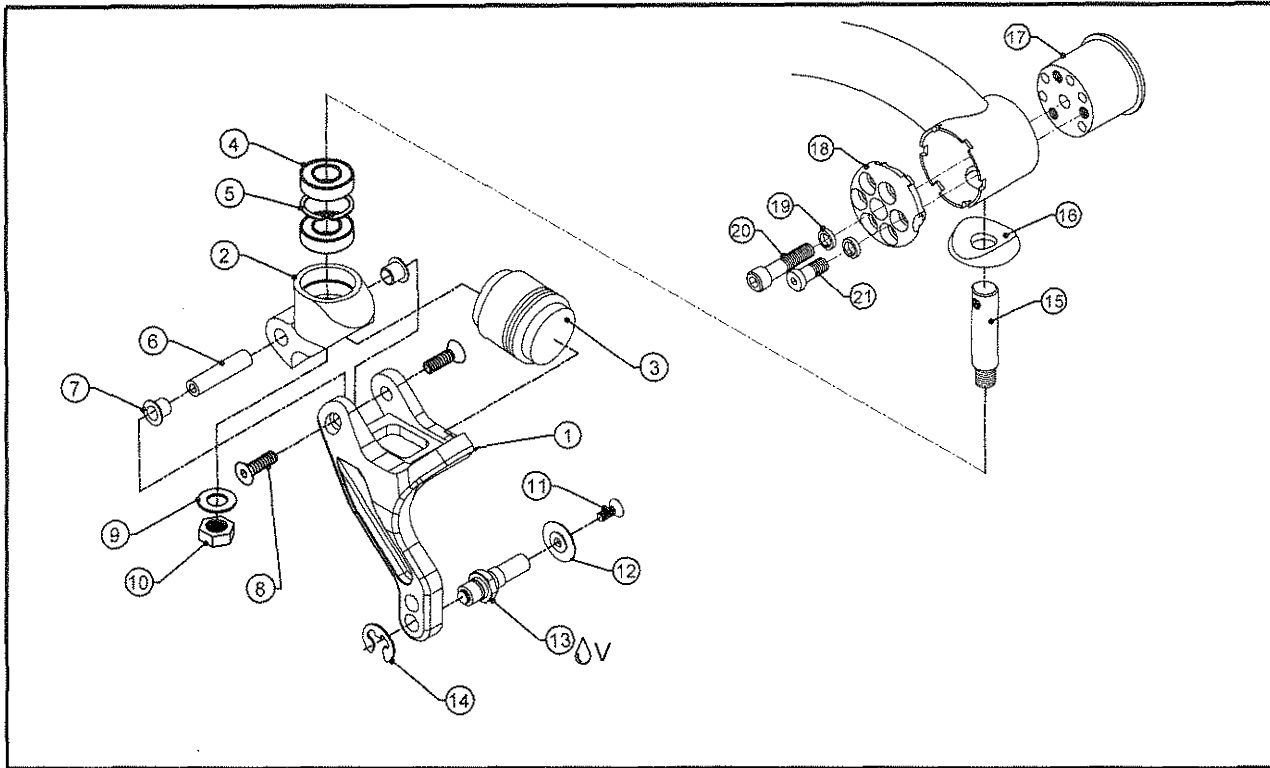

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP107147	GLIDE SUSPENSION FORK SINGLE SIDED LEFT
1B	1	CP107146	GLIDE SUSPENSION FORK SINGLE SIDED RIGHT
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW WITH THREAD PATCH
11B	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
12A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
12B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
13	1	CP104308	SINGLE FORK AXLE
14	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
15A	1	CP104292	BEARING FORK STEM
15B	1	CP104302	BEARING FORK STEM W/ 1/2" EXT
15C	1	CP104303	BEARING FORK STEM W/ 1" EXT
15D	1	CP104372	TITANIUM BEARING FORK STEM
16A	1	CP105482	CASTER CAP
16B	1	CP105483	CASTER CAP 1/2" EXT
16C	1	CP105484	CASTER CAP 1" EXT
17A	1	CP104298	CASTER MOUNT, RIGHT
17B	1	CP104299	CASTER MOUNT, LEFT
18A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
18B	1	CP104295	Z2 CASTER ENDCAP, LEFT
19	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
20	1	HD100676	1/4-28 X 1-7/16" W/ 3/4" THREAD SHCS, BLACK ZINC
21	1	HD100661	1/4-28 X 29/32" W/ 3/8" THREAD LOW HEAD CAP SCREW, BLACK
1A-3A, 4-11A, 12A-14		AS116100	LIGHT LEFT PRONG CLEAR GLIDE ASSY
1A-3A, 4-11B, 12B-14		AS116159	LIGHT LEFT PRONG BLACK GLIDE ASSY
1B-3A, 4-11A, 12A-14		AS116336	LIGHT RIGHT PRONG CLEAR GLIDE ASSY
1B-3A, 4-11B, 12B-14		AS116371	LIGHT RIGHT PRONG BLACK GLIDE ASSY
1A-3B, 4-11A, 12A-14		AS116195	MED LEFT PRONG CLEAR GLIDE ASSY
1A-3B, 4-11B, 12B-14		AS116230	MED LEFT PRONG BLACK GLIDE ASSY
1B-3B, 4-11A, 12A-14		AS116406	MED RIGHT PRONG CLEAR GLIDE ASSY
1B-3B, 4-11B, 12B-14		AS116441	MED RIGHT PRONG BLACK GLIDE ASSY
1A-3C, 4-11A, 12A-14		AS116265	HEAVY LEFT PRONG CLEAR GLIDE ASSY
1A-3C, 4-11B, 12B-14		AS116300	HEAVY LEFT PRONG BLACK GLIDE ASSY
1B-3C, 4-11A, 12A-14		AS116476	HEAVY RIGHT PRONG CLEAR GLIDE ASSY
1B-3C, 4-11B, 12B-14		AS116511	HEAVY RIGHT PRONG BLACK GLIDE ASSY

GLIDE FORK ASSEMBLY

TiLite™ Aero X, 2GX, Aero R

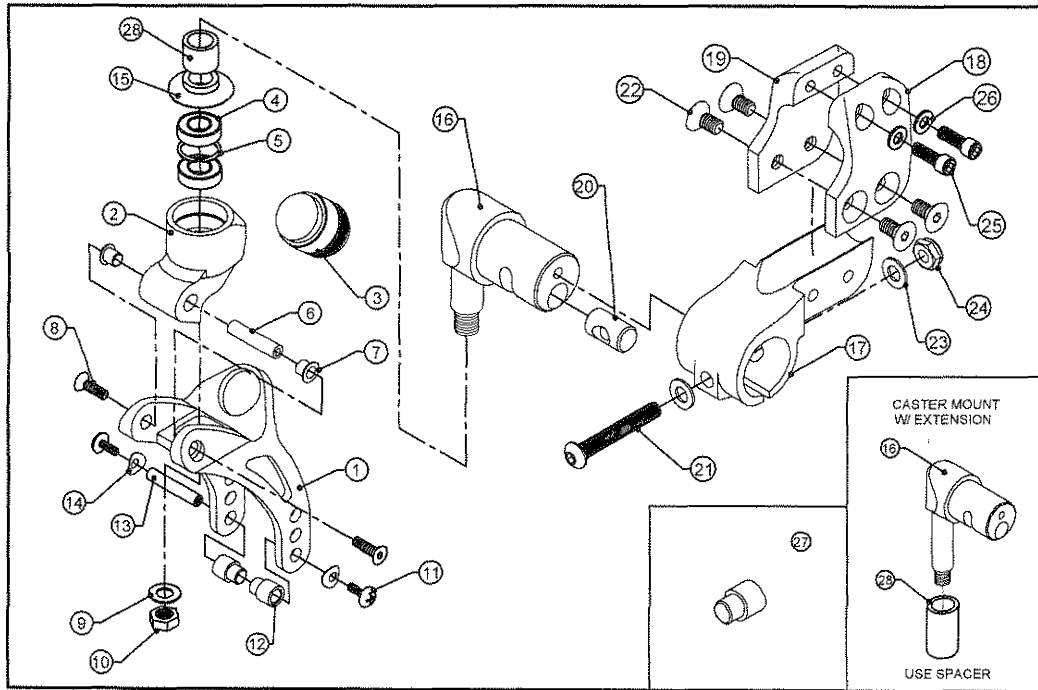

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP107149	GLIDE SUSPENSION FORK BODY
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8" O.D. X 5/16" W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	SUSPENSION FORK BUSHING
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
12A	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
12B	1	CP104405	POLY CASTER SPACER (NON-DISHED SIDE)
12C	1	CP104406	POLY CASTER SPACER (DISHED SIDE)
13	1	CP103041	CASTER AXLE LONG
14	2	CP104375	C SPRING WASHER
15	1	CP102048	CASTER CAP BEARING FORK
16A	1	AS103237	BEARING FORK CASTER MOUNT ASSEMBLY
16B	1	AS103238	BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
16C	1	AS103239	BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
17	1	CP103185	CASTER MOUNT
18A	1	CP103895	CASTER MOUNT CLAMP, COUNTERBORED, RIGHT
18B	1	CP103896	CASTER MOUNT CLAMP, COUNTERBORED, LEFT
19A	1	CP103897	CASTER MOUNT CLAMP, TAPPED, RIGHT
19B	1	CP103898	CASTER MOUNT CLAMP, TAPPED, LEFT
20	1	CP102142	CASTER MOUNT DOWEL Ø.437
21	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
22	4	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
23	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
24	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
25	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
26	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
27A	1	CP104403	POLY CASTER SPACER (NON-DISHED SIDE)
27B	1	CP104404	POLY CASTER SPACER (DISHED SIDE)
28A	1	CP100346	CASTER EXTENSION 1/2"
28B	1	CP100347	CASTER EXTENSION 1"
12A		AS116546	CASTER AXLE SPACERS
12B, 12C		AS116547	POLY CASTER SPACERS
1-3A, 4-12A, 13-14		AS116548	LIGHT AXLE SPACED GLIDE ASSY
1-3B, 4-12A, 13-14		AS116640	MEDIUM AXLE SPACERS GLIDE ASSY
1-3C, 4-12A, 13-14		AS116732	HEAVY AXLE SPACERS GLIDE ASSY
1-3A, 4-12B, 12C-14		AS116594	LIGHT POLY SPACED GLIDE ASSY
1-3B, 4-12B, 12C-14		AS116686	MEDIUM POLY SPACERS GLIDE ASSY
1-3C, 4-12B, 12C-14		AS116778	HEAVY POLY SPACERS GLIDE ASSY



GLIDE FORK ASSEMBLY


TiLite™ Aero X, 2GX, Aero R

TABLE ONE - SPACERS

	SPACER ASSEMBLY
3" x .75" MICRO	AS116546
3" x .75" LIGHT-UP MICRO-CASTER	AS116546
3" x .75" LITESPEED BILLET AL WHEEL W/SOFT ROLL	AS116547
4" x .75" PLASTIC WHEEL W/ POLY	AS116546
4" x .75" LIGHT-UP MICRO-CASTER	AS116546
4" x .75" LITESPEED BILLET AL WHEEL W/POLY	AS116547
4" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
4" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
5" x 1" PLASTIC WHEEL W/ POLY	AS116546
5" x 1.5" LITESPEED PLASTIC WHEEL W/ SOFT ROLL	AS116547
5" x .75" LITESPEED BILLET AL WHEEL W/ POLY	AS116547
5" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
6" x 1" PLASTIC WHEEL W/ POLY	AS116546
6" x 1" LITESPEED BILLET AL WHEEL W/POLY	AS116547
6" x 1.25" LITESPEED BILLET AL WHEEL W/PNEUMATIC	AS116547
6" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
6" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547

GLIDE SINGLE FORK ASSEMBLY


TiLite™ Aero X, 2GX, Aero R

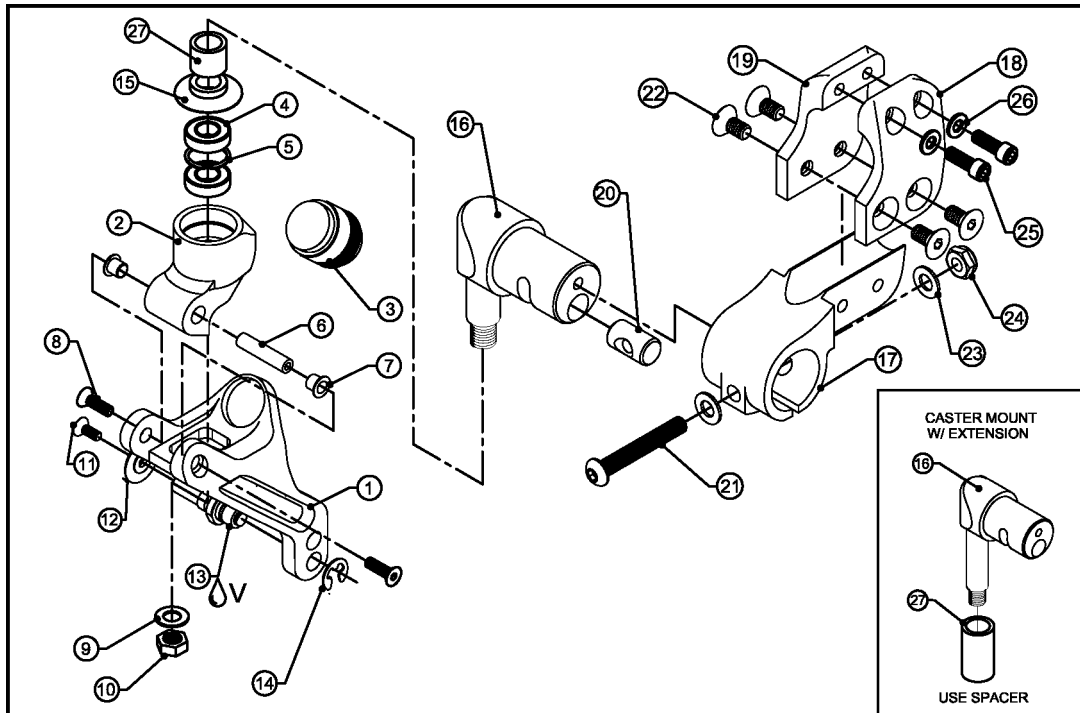

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP107147	GLIDE SUSPENSION FORK SINGLE SIDED LEFT
1B	1	CP107146	GLIDE SUSPENSION FORK SINGLE SIDED RIGHT
2	1	CP107148	SINGLE SIDED SUSPENSION FORK BEARING MOUNT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW WITH THREAD PATCH
11B	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
12A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
12B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
13	1	CP104308	SINGLE FORK AXLE
14	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
15	1	CP102048	CASTER CAP BEARING FORK
16A	1	AS103237	BEARING FORK CASTER MOUNT ASSEMBLY
16B	1	AS103238	BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
16C	1	AS103239	BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
17	1	CP103185	CASTER MOUNT
18A	1	CP103895	CASTER MOUNT CLAMP, COUNTERBORED, RIGHT
18B	1	CP103896	CASTER MOUNT CLAMP, COUNTERBORED, LEFT
19A	1	CP103897	CASTER MOUNT CLAMP, TAPPED, RIGHT
19B	1	CP103898	CASTER MOUNT CLAMP, TAPPED, LEFT
20	1	CP102142	CASTER MOUNT DOWEL Ø.437
21	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
22	4	HD100471	1/4-20 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
23	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
24	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
25	2	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
26	2	HD100348	WASHER .255ID X .395OD X .048THK STEEL, ZINC PLATED
27A	1	CP100346	CASTER EXTENSION 1/2"
27B	1	CP100347	CASTER EXTENSION 1"
1A-3A,4-11A,12A-14		AS116100	LIGHT LEFT PRONG CLEAR GLIDE ASSY
1A-3A,4-11B,12B-14		AS116159	LIGHT LEFT PRONG BLACK GLIDE ASSY
1B-3A,4-11A,12A-14		AS116336	LIGHT RIGHT PRONG CLEAR GLIDE ASSY
1B-3A,4-11B,12B-14		AS116371	LIGHT RIGHT PRONG BLACK GLIDE ASSY
1A-3B,4-11A,12A-14		AS116195	MED LEFT PRONG CLEAR GLIDE ASSY
1A-3B,4-11B,12B-14		AS116230	MED LEFT PRONG BLACK GLIDE ASSY
1B-3B,4-11A,12A-14		AS116406	MED RIGHT PRONG CLEAR GLIDE ASSY
1B-3B,4-11B,12B-14		AS116441	MED RIGHT PRONG BLACK GLIDE ASSY
1A-3C,4-11A,12A-14		AS116265	HEAVY LEFT PRONG CLEAR GLIDE ASSY
1A-3C,4-11B,12B-14		AS116300	HEAVY LEFT PRONG BLACK GLIDE ASSY
1B-3C,4-11A,12A-14		AS116476	HEAVY RIGHT PRONG CLEAR GLIDE ASSY
1B-3C,4-11B,12B-14		AS116511	HEAVY RIGHT PRONG BLACK GLIDE ASSY

GLIDE FORK ASSEMBLY

HEMI HEIGHT TiLite™ Aero X, 2GX

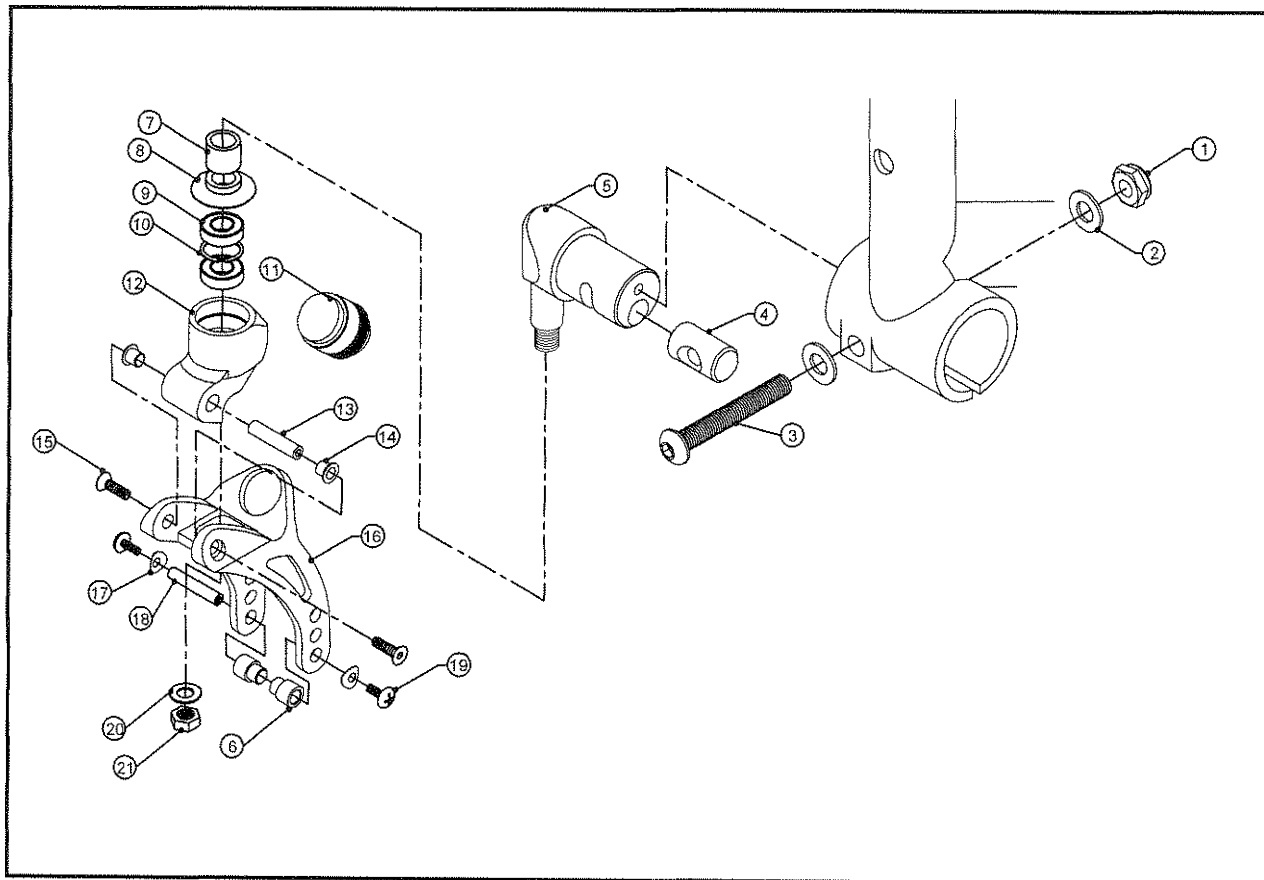

ITEM	QTY	PART NO.	DESCRIPTION
1	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
2	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
3	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
4	1	CP102142	CASTER MOUNT DOWEL DIA. .437
5A	1	AS111159	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY
5B	1	AS111160	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
5C	1	AS111161	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
6A	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
6B	1	CP104405	POLY CASTER SPACER (NON-DISHED SIDE)
6C	1	CP104406	POLY CASTER SPACER (DISHED SIDE)
7A	1	CP100346	CASTER EXTENSION 1/2"
7B	1	CP100347	CASTER EXTENSION 1"
8	1	CP102048	CASTER CAP BEARING FORK
9	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
10	1	HD100330	1-1/8" INTERNAL RETAINING RING
11A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
11B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
11C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
12	1	CP107150	GLIDE SUSPENSION FORK PIVOT
13	1	CP107216	PIVOT AXLE
14	2	CP107217	SUSPENSION FORK BUSHING
15	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
16	1	CP107149	GLIDE SUSPENSION FORK BODY
17	2	CP104375	C SPRING WASHER
18	1	CP103041	CASTER AXLE LONG
19	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC, THREAD PATCH
20	1	HD100353	3/8" AN WASHER STAINLESS STEEL
21	1	HD100352	38-24 NYLOCK NUT
6A		AS116546	CASTER AXLE SPACERS
6B,6C		AS116547	POLY CASTER SPACERS
6A-11A,12-21		AS116548	LIGHT AXLE SPACED GLIDE ASSY
6A-11B,12-21		AS116640	MEDIUM AXLE SPACERS GLIDE ASSY
6A-11C,12-21		AS116732	HEAVY AXLE SPACERS GLIDE ASSY
6B,6C-11A,12-21		AS116594	LIGHT POLY SPACED GLIDE ASSY
6B,6C-11B,12-21		AS116686	MEDIUM POLY SPACERS GLIDE ASSY
6B,6C-11C,12-21		AS116778	HEAVY POLY SPACERS GLIDE ASSY



GLIDE FORK ASSEMBLY

HEMI HEIGHT TiLite™ AERO X, 2GX


TABLE ONE - SPACERS

	SPACER ASSEMBLY
3" x .75" MICRO	AS116546
3" x .75" LIGHT-UP MICRO-CASTER	AS116546
3" x .75" LITESPEED BILLET AL WHEEL W/SOFT ROLL	AS116547
4" x .75" PLASTIC WHEEL W/ POLY	AS116546
4" x .75" LIGHT-UP MICRO-CASTER	AS116546
4" x .75" LITESPEED BILLET AL WHEEL W/POLY	AS116547
4" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
4" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
5" x 1" PLASTIC WHEEL W/ POLY	AS116546
5" x 1.5" LITESPEED PLASTIC WHEEL W/ SOFT ROLL	AS116547
5" x .75" LITESPEED BILLET AL WHEEL W/ POLY	AS116547
5" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
6" x 1" PLASTIC WHEEL W/ POLY	AS116546
6" x 1" LITESPEED BILLET AL WHEEL W/POLY	AS116547
6" x 1.25" LITESPEED BILLET AL WHEEL W/PNEUMATIC	AS116547
6" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
6" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547



GLIDE SINGLE FORK ASSEMBLY

HEMI HEIGHT TiLite™ Aero X, 2GX

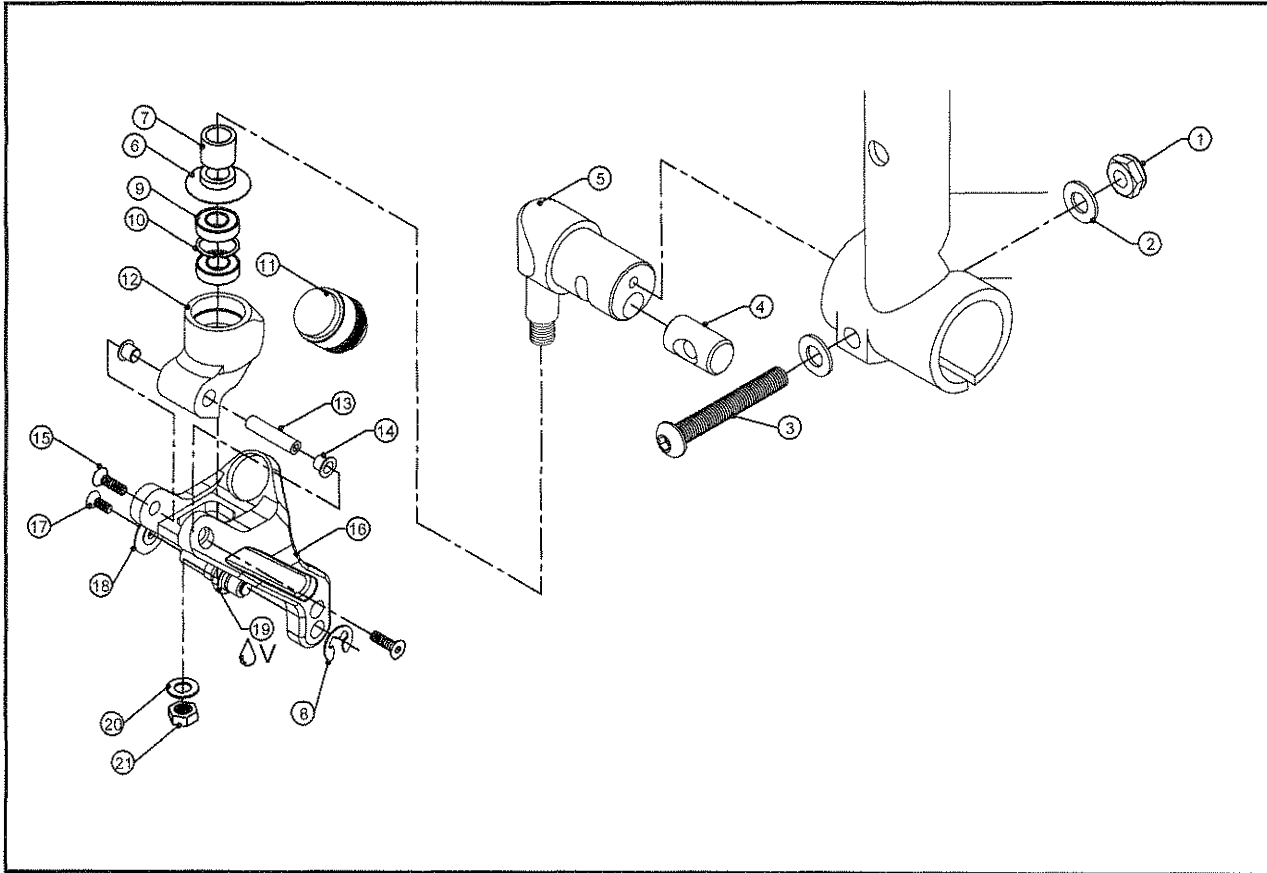


ITEM	QTY	PART NO.	DESCRIPTION
1	1	HD100494	1/4-28 NYLOCK NUT, BLACK ZINC PLATE
2	2	HD100490	1/4" AN WASHER .260 I.D. .500 O.D. .0625 THICK, BLACK ZINC
3	1	HD100440	1/4-28 X 1-3/4" BUTTON HEAD CAP SCREW, BLACK
4	1	CP102142	CASTER MOUNT DOWEL DIA. .437
5A	1	AS111159	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY
5B	1	AS111160	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1/2" EXT
5C	1	AS111161	HEMI HEIGHT BEARING FORK CASTER MOUNT ASSEMBLY 1" EXT
6	1	CP102048	CASTER CAP BEARING FORK
7A	1	CP100346	CASTER EXTENSION 1/2"
7B	1	CP100347	CASTER EXTENSION 1"
8	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
9	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
10	1	HD100330	1-1/8" INTERNAL RETAINING RING
11A	1	CP107213	LIGHT SUSPENSION ELASTOMER
11B	1	CP107214	MEDIUM SUSPENSION ELASTOMER
11C	1	CP107215	HEAVY SUSPENSION ELASTOMER
12	1	CP107150	GLIDE SUSPENSION FORK PIVOT
13	1	CP107216	PIVOT AXLE
14	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
15	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
16A	1	CP107147	GLIDE SUSPENSION FORK SINGLE SIDED LEFT
16B	1	CP107146	GLIDE SUSPENSION FORK SINGLE SIDED RIGHT
17A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW WITH THREAD PATCH
17B	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
18A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
18B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
19	1	CP104308	SINGLE FORK AXLE WITH THREAD PATCH
20	1	HD100353	3/8" AN WASHER STAINLESS STEEL
21	1	HD100352	38-24 NYLOCK NUT
8-11A, 12-16A, 15-17A, 18A-21		AS116100	LIGHT LEFT PRONG CLEAR GLIDE ASSY
8-11A, 12-16A, 15-17B, 18B-21		AS116159	LIGHT LEFT PRONG BLACK GLIDE ASSY
8-11A, 12-16B, 15-17A, 18A-21		AS116336	LIGHT RIGHT PRONG CLEAR GLIDE ASSY
8-11A, 12-16B, 15-17B, 18B-21		AS116371	LIGHT RIGHT PRONG BLACK GLIDE ASSY
8-11B, 12-16A, 15-17A, 18A-21		AS116195	MED LEFT PRONG CLEAR GLIDE ASSY
8-11B, 12-16A, 15-17B, 18B-21		AS116230	MED LEFT PRONG BLACK GLIDE ASSY
8-11B, 12-16B, 15-17A, 18A-21		AS116406	MED RIGHT PRONG CLEAR GLIDE ASSY
8-11B, 12-16B, 15-17B, 18B-21		AS116441	MED RIGHT PRONG BLACK GLIDE ASSY
8-11C, 12-16A, 15-17A, 18A-21		AS116265	HEAVY LEFT PRONG CLEAR GLIDE ASSY
8-11C, 12-16A, 15-17B, 18B-21		AS116300	HEAVY LEFT PRONG BLACK GLIDE ASSY
8-11C, 12-16B, 15-17A, 18A-21		AS116476	HEAVY RIGHT PRONG CLEAR GLIDE ASSY
8-11C, 12-16B, 15-17B, 18B-21		AS116511	HEAVY RIGHT PRONG BLACK GLIDE ASSY



GLIDE FORK ASSEMBLY

TiLite™ Aero X2

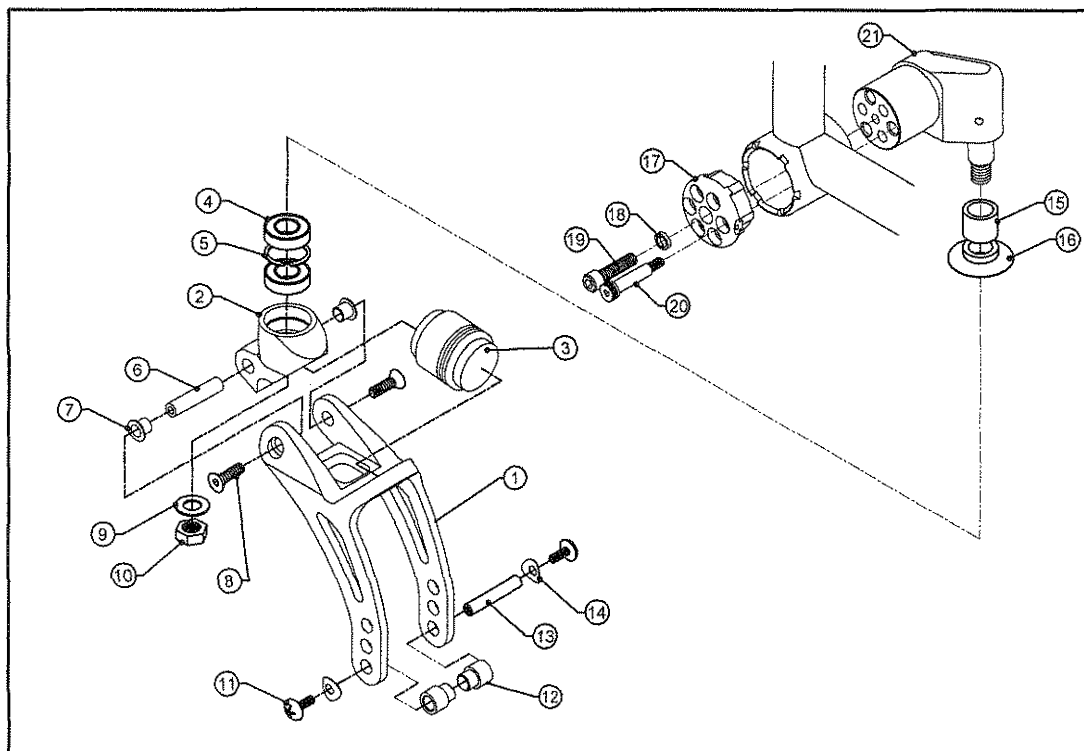

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP107149	GLIDE SUSPENSION FORK BODY
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	SUSPENSION FORK BUSHING
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
12A	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
12B	1	CP104405	POLY CASTER SPACER (NON-DISHED SIDE)
12C	1	CP104406	POLY CASTER SPACER (DISHED SIDE)
13	1	CP103041	CASTER AXLE LONG
14	2	CP104375	C SPRING WASHER
15A	1	CP100346	CASTER EXTENSION 1/2"
15B	1	CP100347	CASTER EXTENSION 1"
16	1	CP102048	CASTER CAP BEARING FORK
17A	1	CP107413	CASTER ENDCAP BIG RIGHT
17B	1	CP107414	CASTER ENDCAP BIG LEFT
18	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
19	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
20	1	HD100724	1/4" X 1" SHOULDER BOLT
21A	1	AS117267	CASTER MOUNT W/STD STEM, RIGHT
21B	1	AS117268	CASTER MOUNT W/1/2" EXT, RIGHT
21C	1	AS117269	CASTER MOUNT W/1" EXT, RIGHT
21D	1	AS117270	CASTER MOUNT W/STD STEM, LEFT
21E	1	AS117271	CASTER MOUNT W/1/2" EXT, LEFT
21F	1	AS117272	CASTER MOUNT W/1" EXT, LEFT
12A		AS116546	CASTER AXLE SPACERS
12B, 12C		AS116547	POLY CASTER SPACERS
1-3A, 4-12A, 13-14		AS116548	LIGHT AXLE SPACED GLIDE ASSY
1-3B, 4-12A, 13-14		AS116640	MEDIUM AXLE SPACERS GLIDE ASSY
1-3C, 4-12A, 13-14		AS116732	HEAVY AXLE SPACERS GLIDE ASSY
1-3A, 4-12B, 12C-14		AS116594	LIGHT POLY SPACED GLIDE ASSY
1-3B, 4-12B, 12C-14		AS116686	MEDIUM POLY SPACERS GLIDE ASSY
1-3C, 4-12B, 12C-14		AS116778	HEAVY POLY SPACERS GLIDE ASSY



GLIDE FORK ASSEMBLY


TiLite™ AERO X2

TABLE ONE - SPACERS

	SPACER ASSEMBLY
3" x .75" MICRO	AS116546
3" x .75" LIGHT-UP MICRO-CASTER	AS116546
3" x .75" LITESPEED BILLET AL WHEEL W/SOFT ROLL	AS116547
4" x .75" PLASTIC WHEEL W/ POLY	AS116546
4" x .75" LIGHT-UP MICRO-CASTER	AS116546
4" x .75" LITESPEED BILLET AL WHEEL W/POLY	AS116547
4" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
4" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
5" x 1" PLASTIC WHEEL W/ POLY	AS116546
5" x 1.5" LITESPEED PLASTIC WHEEL W/ SOFT ROLL	AS116547
5" x .75" LITESPEED BILLET AL WHEEL W/ POLY	AS116547
5" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
6" x 1" PLASTIC WHEEL W/ POLY	AS116546
6" x 1" LITESPEED BILLET AL WHEEL W/POLY	AS116547
6" x 1.25" LITESPEED BILLET AL WHEEL W/PNEUMATIC	AS116547
8" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
8" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547

GLIDE SINGLE FORK ASSEMBLY


TiLite™ AERO X (SERIES 2)

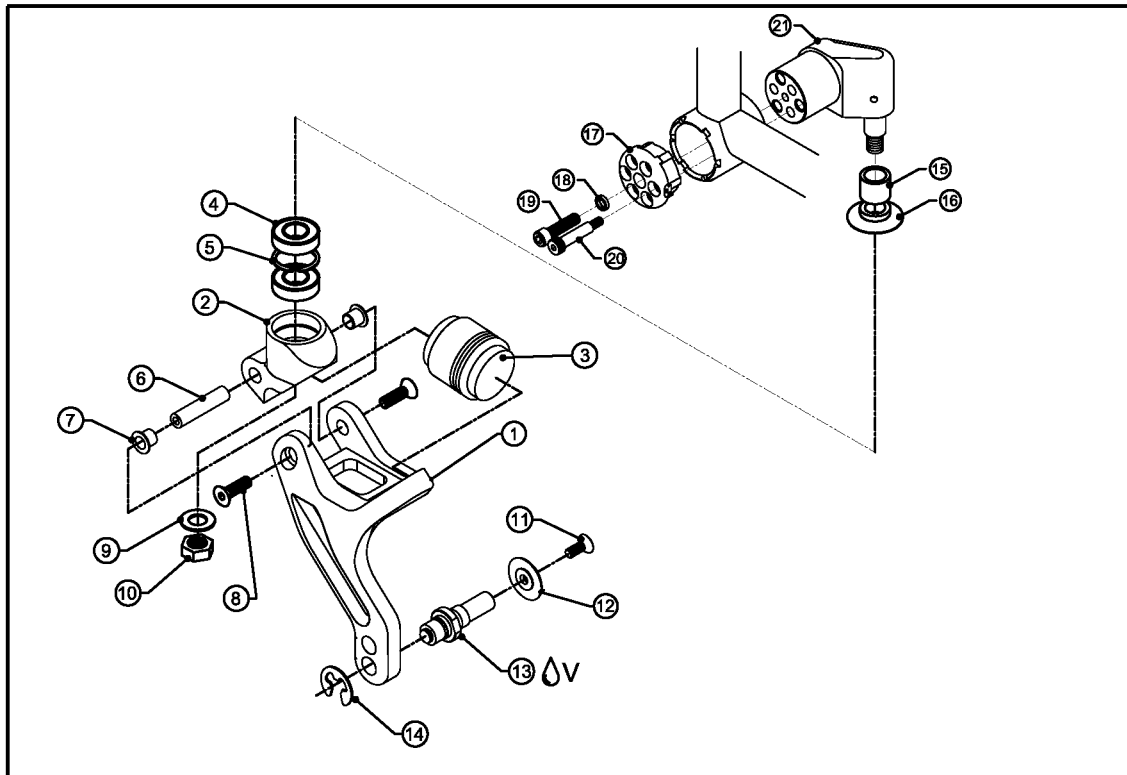

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP107147	GLIDE SUSPENSION FORK SINGLE SIDED LEFT
1B	1	CP107146	GLIDE SUSPENSION FORK SINGLE SIDED RIGHT - (SHOWN)
2	1	CP107148	SINGLE SIDED SUSPENSION FORK BEARING MOUNT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW WITH THREAD PATCH
11B	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
12A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
12B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
13	1	CP104308	SINGLE FORK AXLE
14	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
15A	1	CP100346	CASTER EXTENSION 1/2"
15B	1	CP100347	CASTER EXTENSION 1"
16	1	CP102048	CASTER CAP BEARING FORK
17A	1	CP107413	CASTER ENDCAP BIG RIGHT
17B	1	CP107414	CASTER ENDCAP BIG LEFT
18	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
19	1	HD100332	1/4-20 X 1" SOCKET HEAD CAP SCREW, BLACK ZINC PLATED
20	1	HD100724	1/4" X 1" SHOULDER BOLT
21A	1	AS117267	CASTER MOUNT W/STD STEM, RIGHT
21B	1	AS117268	CASTER MOUNT W/1/2" EXT. RIGHT
21C	1	AS117269	CASTER MOUNT W/1" EXT. RIGHT
21D	1	AS117270	CASTER MOUNT W/STD STEM, LEFT
21E	1	AS117271	CASTER MOUNT W/1/2" EXT. LEFT
21F	1	AS117272	CASTER MOUNT W/1" EXT. LEFT
1A-3A,4-11A,12A-14		AS116100	LIGHT LEFT PRONG CLEAR GLIDE ASSY
1A-3A,4-11B,12B-14		AS116159	LIGHT LEFT PRONG BLACK GLIDE ASSY
1B-3A,4-11A,12A-14		AS116336	LIGHT RIGHT PRONG CLEAR GLIDE ASSY
1B-3A,4-11B,12B-14		AS116371	LIGHT RIGHT PRONG BLACK GLIDE ASSY
1A-3B,4-11A,12A-14		AS116195	MED LEFT PRONG CLEAR GLIDE ASSY
1A-3B,4-11B,12B-14		AS116230	MED LEFT PRONG BLACK GLIDE ASSY
1B-3B,4-11A,12A-14		AS116406	MED RIGHT PRONG CLEAR GLIDE ASSY
1B-3B,4-11B,12B-14		AS116441	MED RIGHT PRONG BLACK GLIDE ASSY
1A-3C,4-11A,12A-14		AS116265	HEAVY LEFT PRONG CLEAR GLIDE ASSY
1A-3C,4-11B,12B-14		AS116300	HEAVY LEFT PRONG BLACK GLIDE ASSY
1B-3C,4-11A,12A-14		AS116476	HEAVY RIGHT PRONG CLEAR GLIDE ASSY
1B-3C,4-11B,12B-14		AS116511	HEAVY RIGHT PRONG BLACK GLIDE ASSY

GLIDE FORK ASSEMBLY

TiLite™ 2GX (SERIES 2)

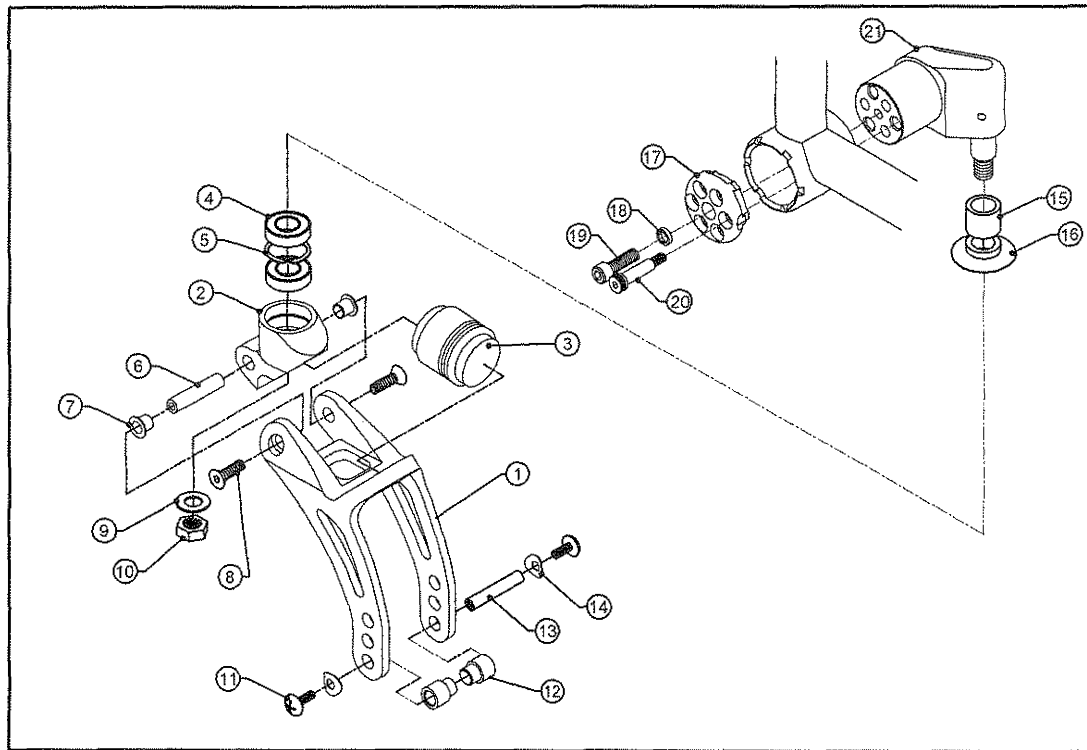

ITEM	QTY	PART NO.	DESCRIPTION
1	1	CP107149	GLIDE SUSPENSION FORK BODY
2	1	CP107150	GLIDE SUSPENSION FORK PIVOT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	SUSPENSION FORK BUSHING
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
12A	2	CP103945	.316ID X .438OD X .364L CASTER AXLE SPACER
12B	1	CP104405	POLY CASTER SPACER (NON-DISHED SIDE)
12C	1	CP104406	POLY CASTER SPACER (DISHED SIDE)
13	1	CP103041	CASTER AXLE LONG
14	2	CP104375	C SPRING WASHER
15A	1	CP100346	CASTER EXTENSION 1/2"
15B	1	CP100347	CASTER EXTENSION 1"
16	1	CP102048	CASTER CAP BEARING FORK
17A	1	CP104294	Z2 CASTER ENDCAP, RIGHT
17B	1	CP104295	Z2 CASTER ENDCAP, LEFT
18	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
19	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
20	1	HD100733	1/4" X 3/4" SHOULDER BOLT
21A	1	AS117267	CASTER MOUNT W/STD STEM, RIGHT
21B	1	AS117268	CASTER MOUNT W/1/2" EXT, RIGHT
21C	1	AS117269	CASTER MOUNT W/1" EXT, RIGHT
21D	1	AS117270	CASTER MOUNT W/STD STEM, LEFT
21E	1	AS117271	CASTER MOUNT W/1/2" EXT, LEFT
21F	1	AS117272	CASTER MOUNT W/1" EXT, LEFT
12A		AS116546	CASTER AXLE SPACERS
12B, 12C		AS116547	POLY CASTER SPACERS
1-3A, 4-12A, 13-14		AS116548	LIGHT AXLE SPACED GLIDE ASSY
1-3B, 4-12A, 13-14		AS116640	MEDIUM AXLE SPACERS GLIDE ASSY
1-3C, 4-12A, 13-14		AS116732	HEAVY AXLE SPACERS GLIDE ASSY
1-3A, 4-12B, 12C-14		AS116594	LIGHT POLY SPACED GLIDE ASSY
1-3B, 4-12B, 12C-14		AS116686	MEDIUM POLY SPACERS GLIDE ASSY
1-3C, 4-12B, 12C-14		AS116778	HEAVY POLY SPACERS GLIDE ASSY



GLIDE FORK ASSEMBLY

TiLite™ 2GX (SERIES 2)


TABLE ONE - SPACERS

	SPACER ASSEMBLY
3" x .75" MICRO	AS116546
3" x .75" LIGHT-UP MICRO-CASTER	AS116546
3" x .75" LITESPEED BILLET AL WHEEL W/SOFT ROLL	AS116547
4" x .75" PLASTIC WHEEL W/ POLY	AS116546
4" x .75" LIGHT-UP MICRO-CASTER	AS116546
4" x .75" LITESPEED BILLET AL WHEEL W/POLY	AS116547
4" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
4" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
5" x 1" PLASTIC WHEEL W/ POLY	AS116546
5" x 1.5" LITESPEED PLASTIC WHEEL W/ SOFT ROLL	AS116547
5" x .75" LITESPEED BILLET AL WHEEL W/ POLY	AS116547
5" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547
6" x 1" PLASTIC WHEEL W/ POLY	AS116546
6" x 1" LITESPEED BILLET AL WHEEL W/POLY	AS116547
6" x 1.25" LITESPEED BILLET AL WHEEL W/PNEUMATIC	AS116547
6" x 1.5" LITESPEED PLASTIC WHEEL W/SOFT ROLL	AS116547
6" x 1.5" LITESPEED BILLET AL WHEEL W/ SOFT ROLL	AS116547



GLIDE SINGLE FORK ASSEMBLY


TiLite™ 2GX (SERIES 2)

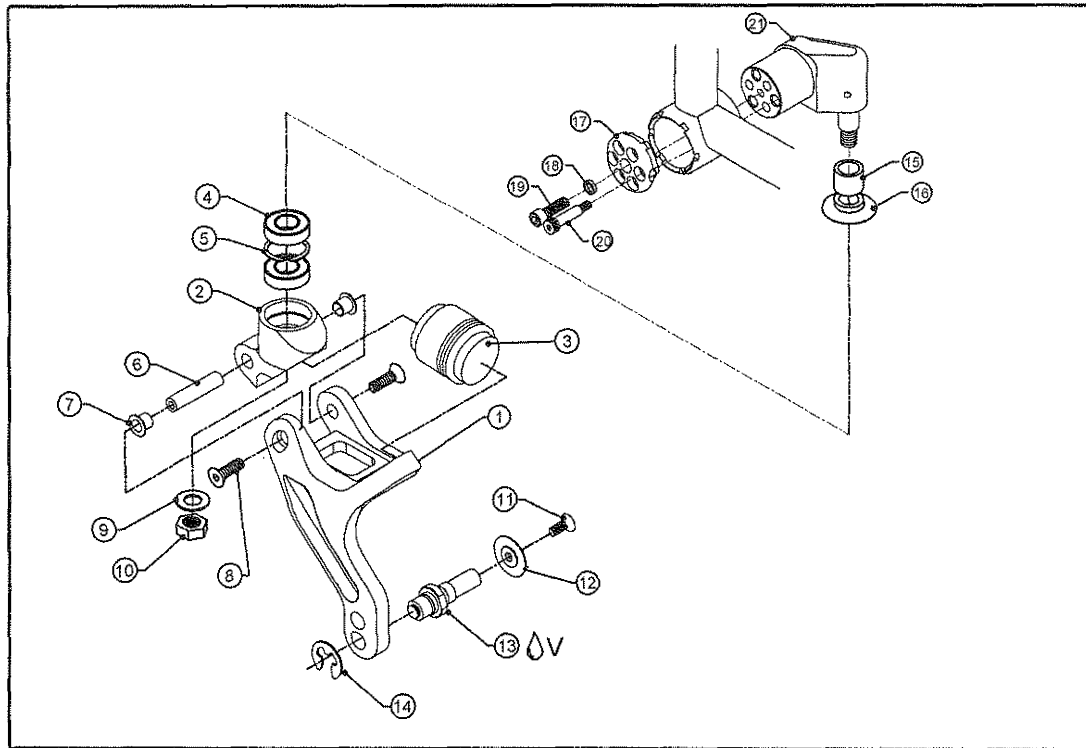

ITEM	QTY	PART NO.	DESCRIPTION
1A	1	CP107147	GLIDE SUSPENSION FORK SINGLE SIDED LEFT
1B	1	CP107146	GLIDE SUSPENSION FORK SINGLE SIDED RIGHT - (SHOWN)
2	1	CP107148	SINGLE SIDED SUSPENSION FORK BEARING MOUNT
3A	1	CP107213	LIGHT SUSPENSION ELASTOMER (0-125 LBS)
3B	1	CP107214	MEDIUM SUSPENSION ELASTOMER (125-180 LBS)
3C	1	CP107215	HEAVY SUSPENSION ELASTOMER (180-265 LBS)
4	2	HD100011	BEARING 1/2" I.D. X 1-1/8 O.D. X 5/16 W
5	1	HD100330	1-1/8" INTERNAL RETAINING RING
6	1	CP107216	PIVOT AXLE
7	2	CP107217	FLANGE BUSHING .375 OD X 5/16 ID X .375 DEPTH
8	2	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
9	1	HD100353	3/8" AN WASHER STAINLESS STEEL
10	1	HD100352	3/8-24 NYLOCK NUT
11A	1	HD100675	10-32 X 5/8" FLAT HEAD CAP SCREW WITH THREAD PATCH
11B	1	HD100339	10-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC, THREAD PATCH
12A	1	CP104309	SINGLE SIDED CASTER CAP, CLEAR ANO
12B	1	CP104370	SINGLE SIDED CASTER CAP, BLACK ANO
13	1	CP104308	SINGLE FORK AXLE
14	1	HD100663	1/4" E-STYLE EXTERNAL RETAINING RING, BLACK ZINC
15A	1	CP100346	CASTER EXTENSION 1/2"
15B	1	CP100347	CASTER EXTENSION 1"
16	1	CP102048	CASTER CAP BEARING FORK
17A	1	CP107413	CASTER ENDCAP BIG RIGHT
17B	1	CP107414	CASTER ENDCAP BIG LEFT
18	2	HD100272	1/4" HIGH COLLAR SPLIT LOCK WASHER
19	1	HD100341	1/4-20 X 3/4" SOCKET HEAD CAP SCREW, BLACK ZINC
20	1	HD100733	1/4" X 3/4" SHOULDER BOLT
21A	1	AS117267	CASTER MOUNT W/STD STEM, RIGHT
21B	1	AS117268	CASTER MOUNT W/1/2" EXT, RIGHT
21C	1	AS117269	CASTER MOUNT W/1" EXT, RIGHT
21D	1	AS117270	CASTER MOUNT W/STD STEM, LEFT
21E	1	AS117271	CASTER MOUNT W/1/2" EXT, LEFT
21F	1	AS117272	CASTER MOUNT W/1" EXT, LEFT
1A-3A, 4-11A, 12A-14		AS116100	LIGHT LEFT PRONG CLEAR GLIDE ASSY
1A-3A, 4-11B, 12B-14		AS116159	LIGHT LEFT PRONG BLACK GLIDE ASSY
1B-3A, 4-11A, 12A-14		AS116336	LIGHT RIGHT PRONG CLEAR GLIDE ASSY
1B-3A, 4-11B, 12B-14		AS116371	LIGHT RIGHT PRONG BLACK GLIDE ASSY
1A-3B, 4-11A, 12A-14		AS116195	MED LEFT PRONG CLEAR GLIDE ASSY
1A-3B, 4-11B, 12B-14		AS116230	MED LEFT PRONG BLACK GLIDE ASSY
1B-3B, 4-11A, 12A-14		AS116406	MED RIGHT PRONG CLEAR GLIDE ASSY
1B-3B, 4-11B, 12B-14		AS116441	MED RIGHT PRONG BLACK GLIDE ASSY
1A-3C, 4-11A, 12A-14		AS116265	HEAVY LEFT PRONG CLEAR GLIDE ASSY
1A-3C, 4-11B, 12B-14		AS116300	HEAVY LEFT PRONG BLACK GLIDE ASSY
1B-3C, 4-11A, 12A-14		AS116476	HEAVY RIGHT PRONG CLEAR GLIDE ASSY
1B-3C, 4-11B, 12B-14		AS116511	HEAVY RIGHT PRONG BLACK GLIDE ASSY

3" & 4" FORK ASSY

TiLite™ X, SX, EVO, TRA, YRA, YG, YGS, YR, TR, TRC, XC

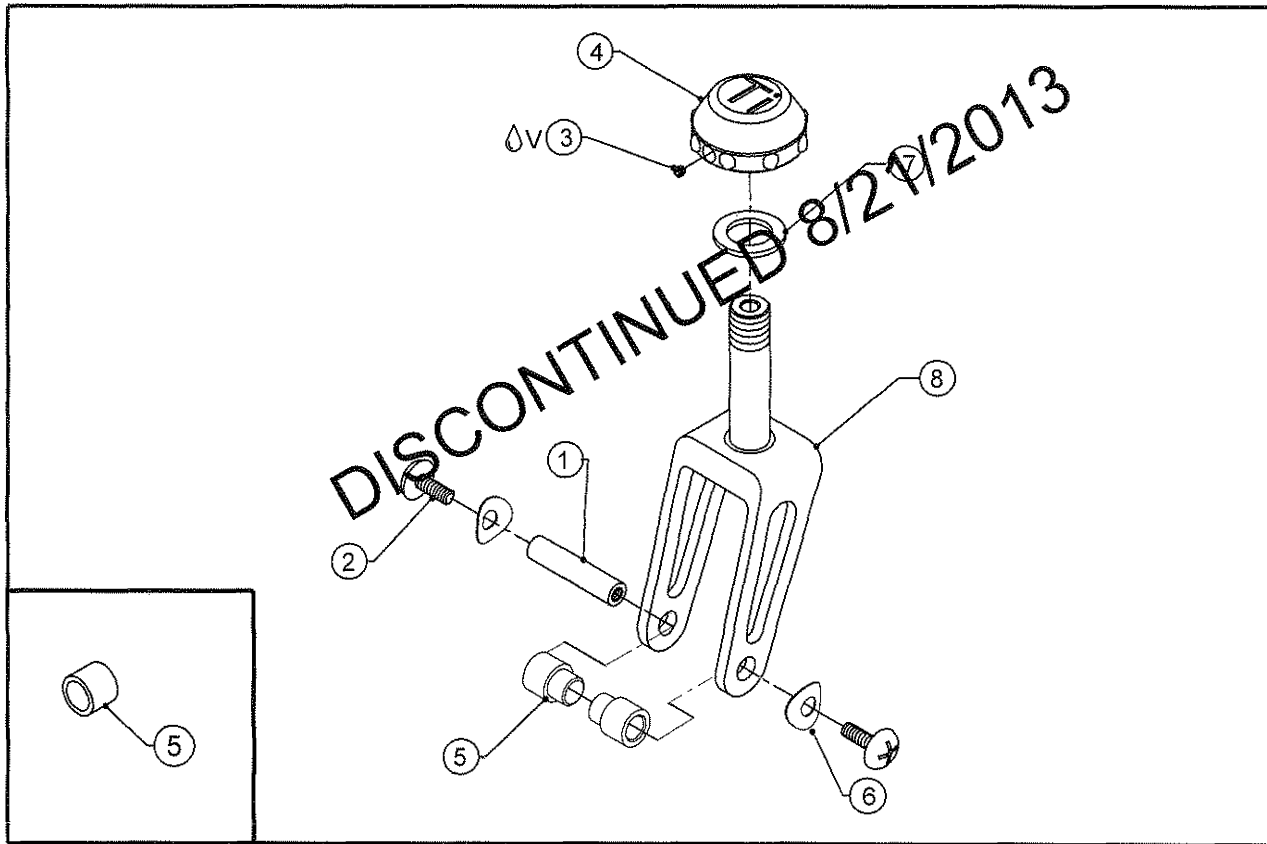


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100005	CASTER AXLE, 3" & 4"
2	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
3	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
4	1	CP100928	CASTER CAP (MOLDED)
5	2	TABLE ONE	CASTER SPACER
6	2	CP104375	C SPRING WASHER
7	1	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06" THK, YELLOW ZINC
8	1	AS104335	3"-4" FORK AND STEM ASSEMBLY
1-8		TABLE ONE	3" & 4" W/ CAP FRONT FORK ASSY

TABLE ONE - SPACERS AND SHORT FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	FORK ASSEMBLY
3" MICRO	CP100007	CP100007	AS101108
3" VOLCANIC	CP100007	CP100007	AS101108
4" PLASTIC W/ POLY	CP100007	CP100007	AS101108
4" VOLCANIC	CP100007	CP100007	AS101108
4" PERFORMANCE W/ POLY	CP105137	CP105138	AS111424
4" PERFORMANCE W/ SOFT ROLL	CP105137	CP105138	AS111424
4" 1.5" WIDE POLY	CP100007	CP100007	AS101108

5"-8" FORK ASSY

TiLite™ X, SX, YRA, TRA, EVO, YG, YGS

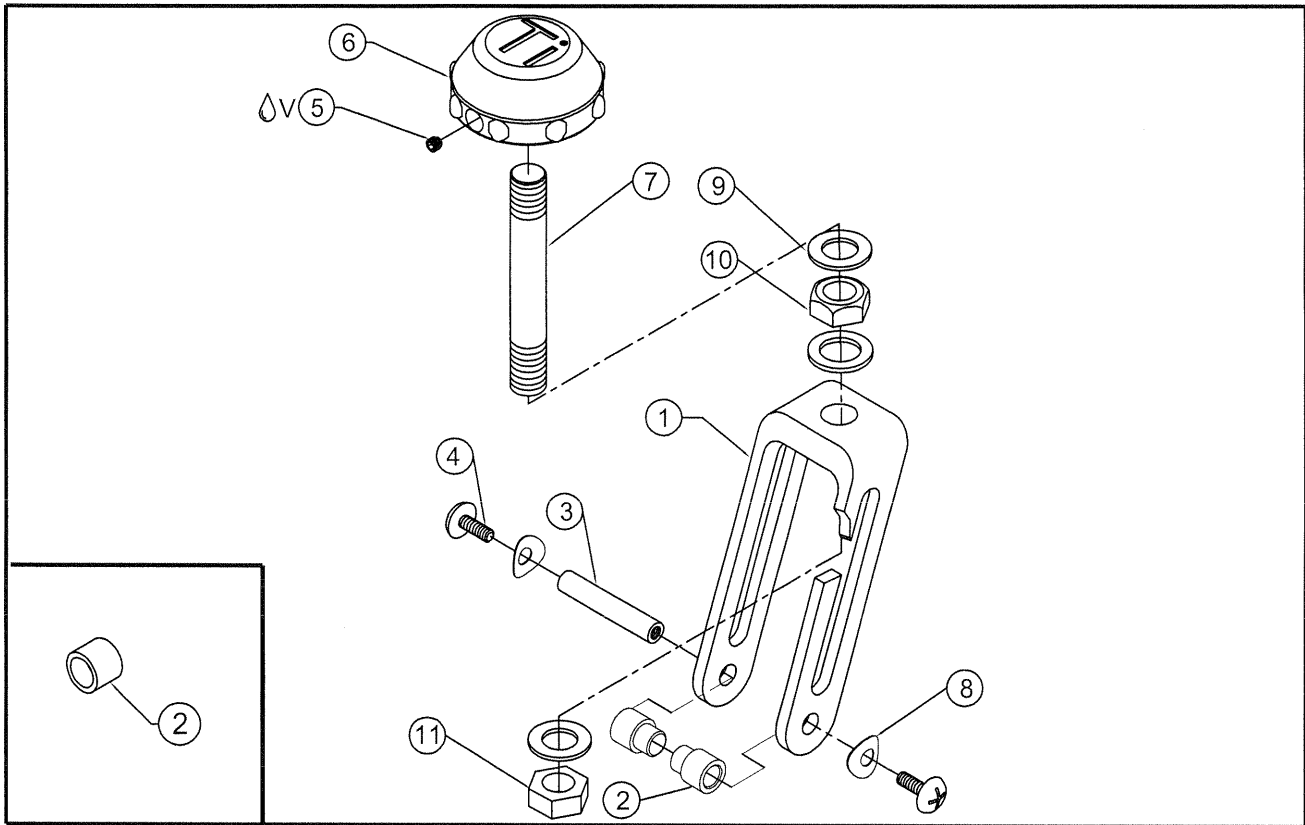


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100335	FORK 5" & 6"
1B	1	CP100339	FORK 7" AND 8"
2	2	TABLE ONE	CASTER SPACER
3	1	HD100085	CASTER AXLE, 5"-8"
4	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
5ΔV	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
6	1	CP100928	CASTER CAP (MOLDED)
7	1	CP101413	CASTER STEM STD W/ 1/2" THREAD
8	2	CP104375	C SPRING WASHER
9	3	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06 THK, YELLOW ZINC
10	1	HD100086	1/2-20 UNF JAM NUT
11	1	HD100087	1/2-20 UNF LOCK NUT THIN
1A, 2-11		AS101096	5" & 6" FORK ASSY
1B, 2-11		AS101097	7" & 8" FORK ASSY

TABLE ONE - SPACERS AND FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	5"-6" FORK	7"-8" FORK
3" MICRO	CP100007	CP100007	AS101114	AS101110
3" VOLCANIC	CP100007	CP100007	AS101114	AS101110
4" PLASTIC W/ POLY	CP100007	CP100007	AS101114	AS101110
4" VOLCANIC	CP100007	CP100007	AS101114	AS101110
4" PERFORMANCE W/ POLY	CP105139	CP105140	AS111425	AS111426
4" PERFORMANCE W/ SOFT ROLL	CP105139	CP105140	AS111425	AS111426
4" 1.5" WIDE POLY	CP100007	CP100007	AS101114	AS101110
5" PLASTIC W/ POLY	HD100084	HD100084	AS101096	AS101097
5" PERFORMANCE W/ POLY	CP105139	CP105140	AS111425	AS111426
5" PERFORMANCE W/ SOFT ROLL	CP105139	CP105140	AS111425	AS111426
5" 1.5" WIDE POLY	HD100084	HD100084	AS101096	AS101097
6" PLASTIC W/ POLY	HD100084	HD100084	AS101096	AS101097
6" PLASTIC W/ PNEUMATIC	HD100084	HD100084	AS101096	AS101097
6" PERFORMANCE W/ POLY	CP105139	CP105140	AS111425	AS111426
6" PERFORMANCE W/ PNEUMATIC	CP105139	CP105140	AS111425	AS111426
6" BILLET W/ SOFT ROLL	HD100084	HD100084	AS101096	AS101097
8" PLASTIC W/ PNEUMATIC	HD100084	HD100084	AS101096	AS101097

5" SPECIAL FORK ASSY


TiLite™ TR, TRC, YR, TRA, YRA, EVO, XC

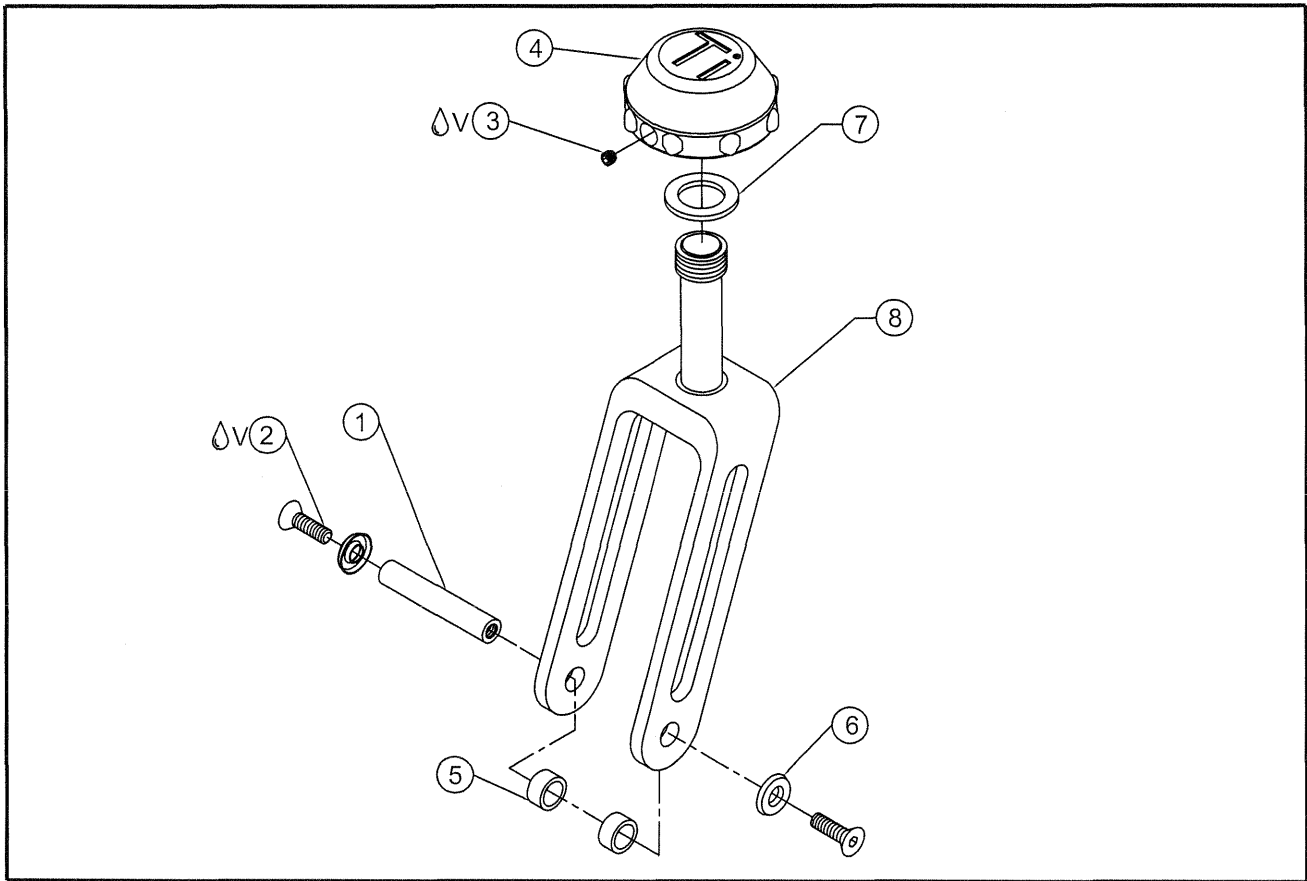

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	HD100085	CASTER AXLE, 5"-8"
2  V	2	HD100063	10-32 X 1/2" FLAT HEAD CAP SCREW
3  V	1	HD100301	8-32 X 3/16" SOCKET SET SCREW
4	1	CP100928	CASTER CAP (MOLDED)
5	2	HD100084	.438 X .220 SPACER
6	2	HD100061	#10 COUNTERSUNK FINISHING WASHER
7	1	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06" THK, YELLOW ZINC
8	1	AS104336	5"-6" FORK AND STEM ASSEMBLY
1-8		AS101094	5" SPECIAL FORK ASSY

5"-8" FORK ASSY W/EXTENSIONS

TiLite™ X, SX, TRA, YRA, EVO, YG, YGS

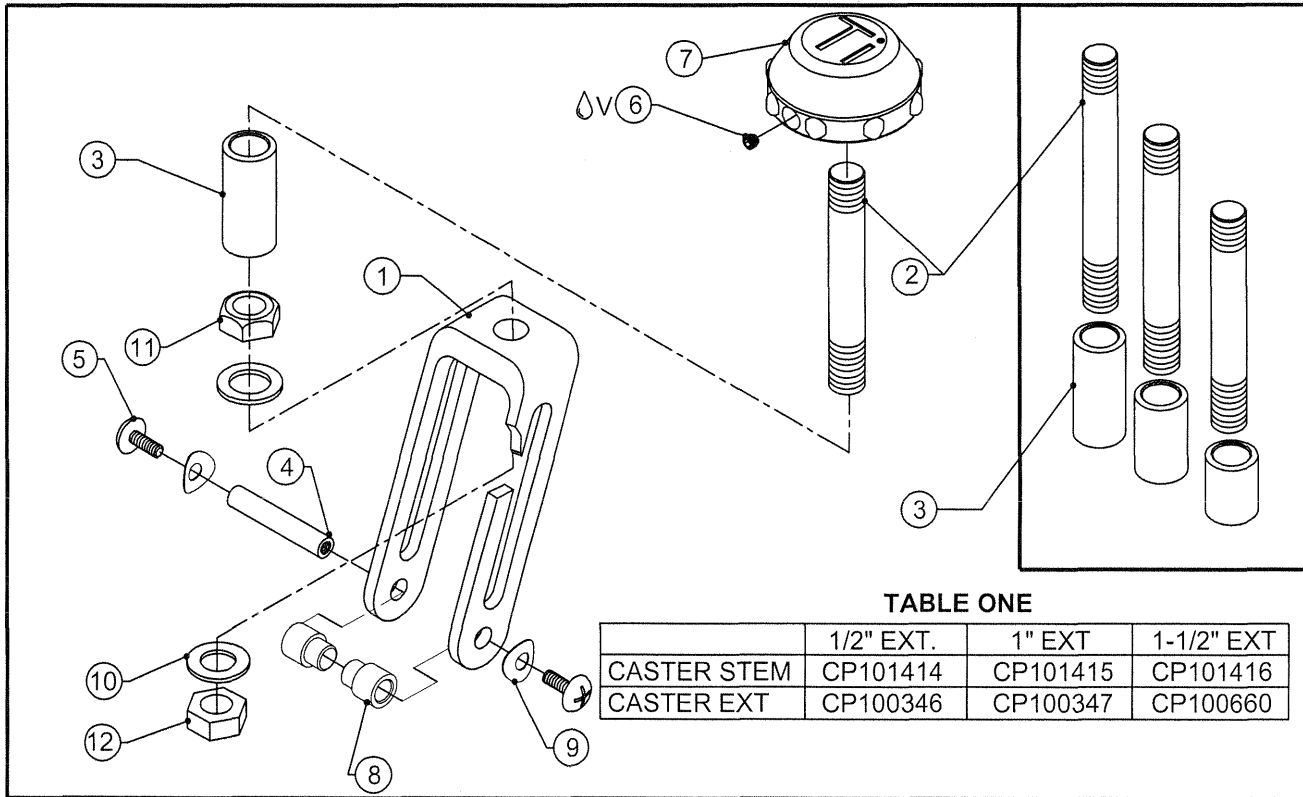


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100335	FORK 5" & 6"
1B	1	CP100339	FORK 7" & 8"
2	1	TABLE ONE	CASTER STEM
3	1	TABLE ONE	CASTER EXTENSION
4	1	HD100085	CASTER AXLE, 5"-8"
5	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
6ΔV	1	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
7	1	CP100928	CASTER CAP (MOLDED)
8	2	TABLE TWO	CASTER SPACER
9	2	CP104375	C SPRING WASHER
10	2	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06 THK, YELLOW ZINC
11	1	HD100086	1/2-20 UNF JAM NUT
12	1	HD100087	1/2-20 UNF LOCK NUT THIN
1A, 2-12		TABLE TWO	5" & 6" W/ 1/2" EXT FORK ASSY
1B, 2-12		TABLE TWO	7" & 8" W/ 1/2" EXT FORK ASSY

TABLE TWO - SPACERS AND FORK ASSEMBLIES

	INSIDE SPACER	OUTSIDE SPACER	5"-6" FORK 1/2" EXT	7"-8" FORK 1/2" EXT	5"-6" FORK 1" EXT	7"-8" FORK 1" EXT	5"-6" FORK 1-1/2" EXT	7"-8" FORK 1-1/2" EXT
3" MICRO	CP100007	CP100007	AS101115	AS101111	AS101116	AS101112	AS101117	AS101113
3" VOLCANIC	CP100007	CP100007	AS101115	AS101111	AS101116	AS101112	AS101117	AS101113
4" PLASTIC W/ POLY	CP100007	CP100007	AS101115	AS101111	AS101116	AS101112	AS101117	AS101113
4" VOLCANIC	CP100007	CP100007	AS101115	AS101111	AS101116	AS101112	AS101117	AS101113
4" PERFORMANCE W/ POLY	CP105139	CP105140	AS111427	AS111428	AS111429	AS111430	AS111431	AS111432
4" PERFORMANCE W/ SOFT ROLL	CP105139	CP105140	AS111427	AS111428	AS111429	AS111430	AS111431	AS111432
4" 1.5" WIDE POLY	CP100007	CP100007	AS101115	AS101111	AS101116	AS101112	AS101117	AS101113
5" PLASTIC W/ POLY	HD100084	HD100084	AS101098	AS101099	AS101100	AS101101	AS101102	AS101103
5" PERFORMANCE W/ POLY	CP105139	CP105140	AS111427	AS111428	AS111429	AS111430	AS111431	AS111432
5" PERFORMANCE W/ SOFT ROLL	CP105139	CP105140	AS111427	AS111428	AS111429	AS111430	AS111431	AS111432
5" 1.5" WIDE POLY	HD100084	HD100084	AS101098	AS101099	AS101100	AS101101	AS101102	AS101103
6" PLASTIC W/ POLY	HD100084	HD100084	AS101098	AS101099	AS101100	AS101101	AS101102	AS101103
6" PLASTIC W/ PNEUMATIC	HD100084	HD100084	AS101098	AS101099	AS101100	AS101101	AS101102	AS101103
6" PERFORMANCE W/ POLY	CP105139	CP105140	AS111427	AS111428	AS111429	AS111430	AS111431	AS111432
6" PERFORMANCE W/ PNEUMATIC	CP105139	CP105140	AS111427	AS111428	AS111429	AS111430	AS111431	AS111432
6" BILLET W/ SOFT ROLL	HD100084	HD100084	AS101098	AS101099	AS101100	AS101101	AS101102	AS101103
8" PLASTIC W/ PNEUMATIC	HD100084	HD100084	N/A	AS101099	N/A	AS101101	N/A	AS101103



FORK ASSY 3" AND 4" CASTER PIN LOCK

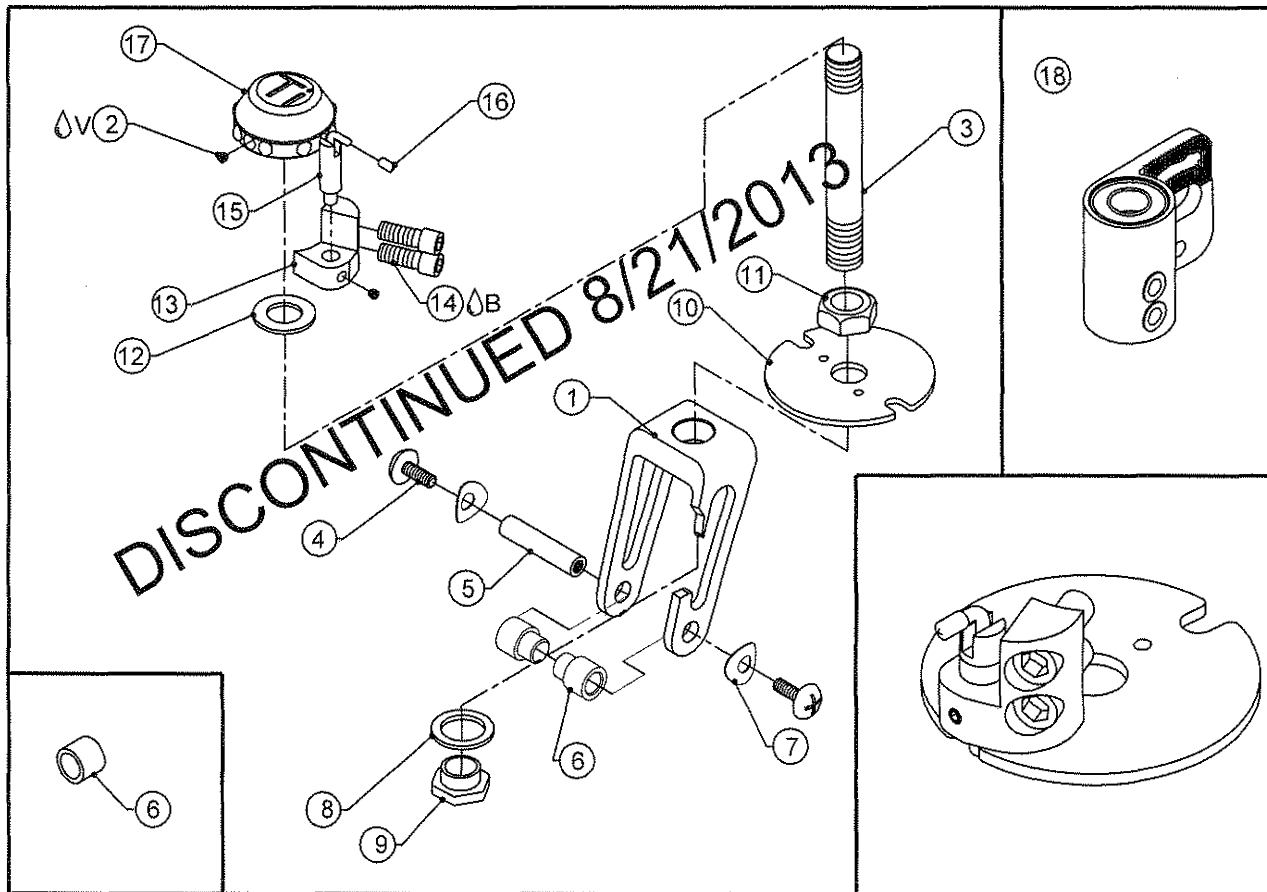

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100760	3" AND 4" CASTER PIN LOCK FORK
2 Δ V	2	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
3	1	CP101413	CASTER STEM STD W/ 1/2" THREAD
4	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
5	1	CP100005	CASTER AXLE, 3" & 4"
6	2	TABLE ONE	CASTER SPACER
7	2	CP104375	C SPRING WASHER
8	1	HD100072	STEEL WASHER, I.D. .630 X .868 O.D. X .036 THICK
9	1	CP100750	1/2-20 JAM NUT, SHOULDERED
10	1	CP100685	LOCK RING, STAINLESS STEEL
11	1	HD100086	1/2-20 UNF JAM NUT
12	1	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06" THK, YELLOW ZINC
13A	1	CP100684	CASTER PIN MOUNT, RIGHT SIDE
13B	1	CP100687	CASTER PIN MOUNT, LEFT SIDE
14 Δ B	2	HD100136	10-32 X 3/8" SOCKET HEAD CAP SCREW, CLEAR ZINC PLATED
15	1	HD100139	L-HANDLE LOCK, 3/8-16 UNC 2A STEEL, CLEAR ZINC PLATE
16	1	HD100137	CAP, VINYL 1/8 I.D. X .50 LG BLACK, 75 DUROMETER HARDNESS
17	1	CP100928	CASTER CAP (MOLDING)
18A	1	AS111116	CASTER PIN LOCK HOUSING RIGHT
18B	1	AS111117	CASTER PIN LOCK HOUSING LEFT


TABLE ONE - SPACERS

	INSIDE SPACER	OUTSIDE SPACER
3" MICRO	CP100007	CP100007
3" VOLCANIC	CP100007	CP100007
4" PLASTIC W/ POLY	CP100007	CP100007
4" VOLCANIC	CP100007	CP100007
4" PERFORMANCE W/ POLY	CP105137	CP105138
4" PERFORMANCE W/ SOFT ROLL	CP105137	CP105138
4" 1.5" WIDE POLY	CP100007	CP100007

* SEE WHEEL ASSEMBLIES AND FORK ASSEMBLIES FOR INDIVIDUAL COMPONENTS

FORK ASSY 5" SPECIAL CASTER PIN LOCK


TiLite™ TR, TRC, YR

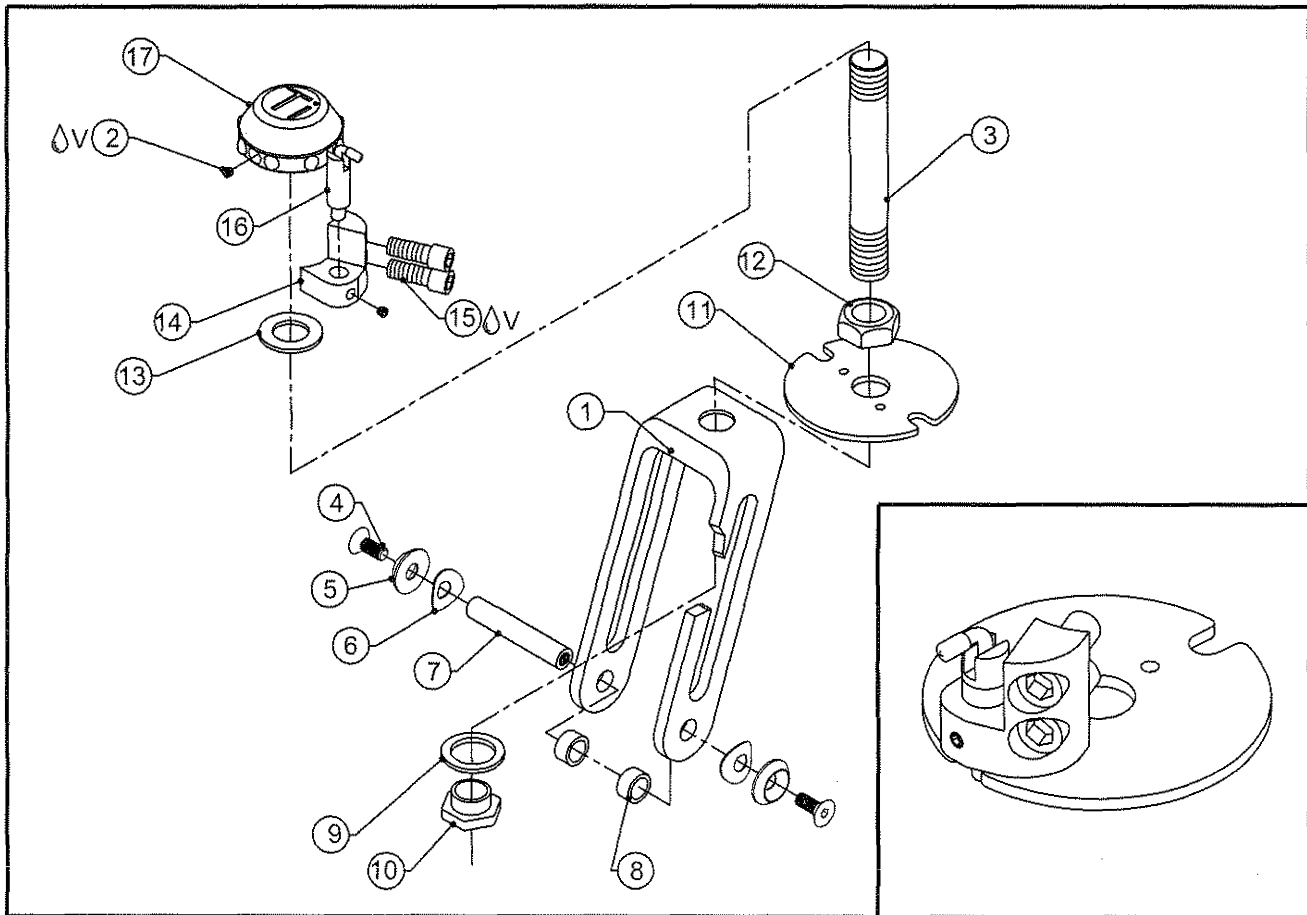

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP101199	5" SPECIAL CASTER PIN LOCK FORK
2 Δ V	2	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
3A	1	CP100924	FORK STEM STD W/ 1/2" THREAD
3B	1	CP100925	FORK STEM STD W/ 1/2" THREAD TI
4	2	HD100536	10-32 X 1/2" FLAT HEAD CAP SCREW, BLACK ZINC
5	2	CP104362	#10 CONICAL WASHER
6	2	CP104375	C SPRING WASHER
7	1	HD100085	CASTER AXLE, 5" & 6"
8	2	HD100084	.438 X .220 SPACER
9	1	HD100072	STEEL WASHER, I.D. .630 X .868 O.D. X .036 THICK
10	1	CP100750	1/2-20 JAM NUT, SHOULDERED
11	1	CP100685	LOCK RING, STAINLESS STEEL
12	1	HD100086	1/2-20 UNF JAM NUT
13	1	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06" THK, YELLOW ZINC
14A	1	CP100684	CASTER PIN MOUNT, RIGHT SIDE
14B	1	CP100687	CASTER PIN MOUNT, LEFT SIDE
15 Δ V	2	HD100136	10-32 X 3/8" SOCKET HEAD CAP SCREW, CLEAR ZINC PLATED
16	2	HD100139	L-HANDLE LOCK, 3/8-16 UNC 2A STEEL, CLEAR ZINC PLATE
17	1	CP100928	CASTER CAP (MOLDED)

* SEE WHEEL ASSEMBLIES AND FORK ASSEMBLIES FOR INDIVIDUAL COMPONENTS
CON = CONSULT WITH FACTORY FOR DRILLING HOLES FOR PIN LOCK PLACEMENT

FORK ASSY 5" - 8" CASTER PIN LOCK

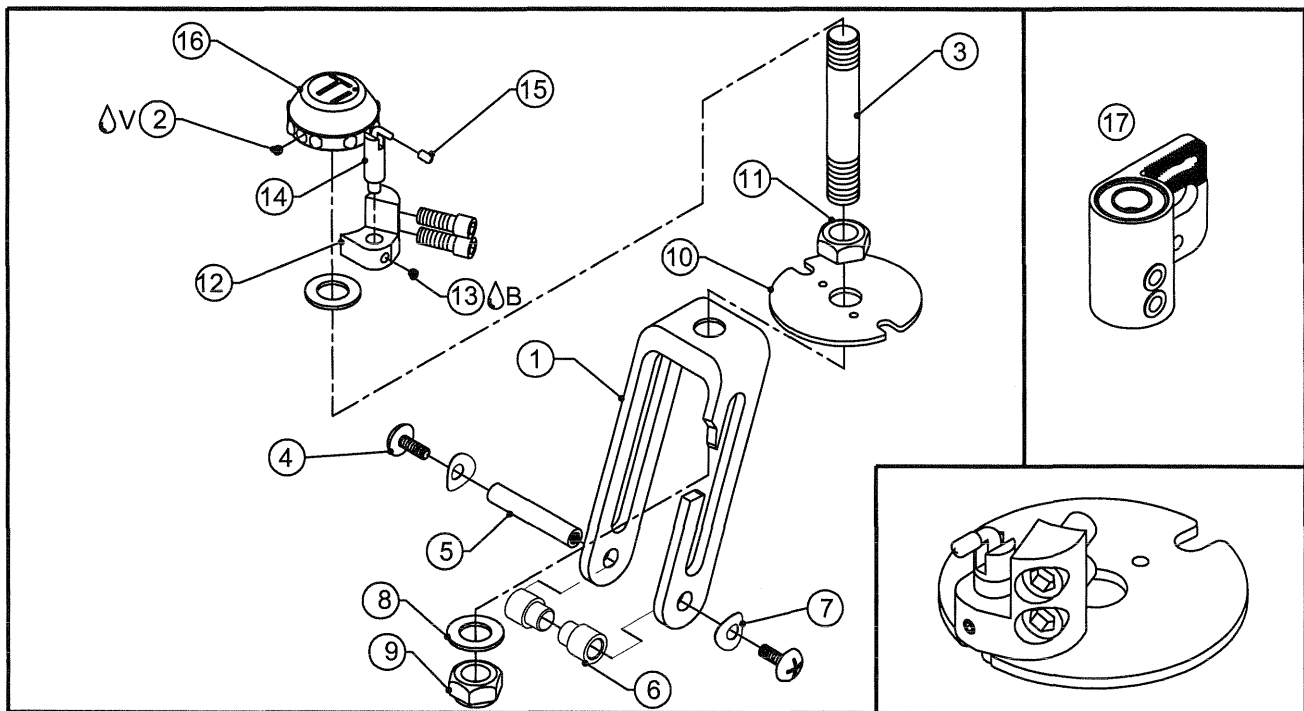


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100335	FORK 5" & 6"
1B	1	CP100339	FORK 7" & 8"
2 Δ V	2	HD100301	8-32 X 3/16" SOCKET SET SCREW, STAINLESS STEEL
3	1	CP100924	FORK STEM STD W/ 1/2" THREADED
4	2	HD100331	10-32 X 3/8" TRUSS HEAD SCREW, BLACK ZINC PLATE
5	1	HD100085	CASTER AXLE, 5" & 6"
6	2	TABLE ONE	CASTER SPACER
7	2	CP104375	C SPRING WASHER
8	2	HD100007	1/2" AN WASHER, .500" I.D. X .875" O.D. X .06" THK, YELLOW ZINC
9	1	HD100087	1/2-20 UNF LOCK NUT THIN
10	1	CP100685	LOCK RING, STAINLESS STEEL
11	1	HD100086	1/2-20 UNF JAM NUT
12A	1	CP100684	CASTER PIN MOUNT, RIGHT SIDE
12B	1	CP100687	CASTER PIN MOUNT, LEFT SIDE
13 Δ B	2	HD100136	10-32 X 3/8" SOCKET HEAD CAP SCREW, CLEAR ZINC PLATED
14	2	HD100139	L-HANDLE LOCK, 3/8-16 UNC 2A STEEL, CLEAR ZINC PLATE
15	2	HD100139	CAP, VINYL 1/8 I.D. X .50 LG BLACK, 75 DUROMETER HARDNESS
16	1	CP100928	CASTER CAP (MOLDED)
17A	1	AS111116	CASTER PIN LOCK HOUSING RIGHT
17B	1	AS111117	CASTER PIN LOCK HOUSING LEFT

TABLE ONE - SPACERS

	INSIDE SPACER	OUTSIDE SPACER
3" MICRO	CP100007	CP100007
3" VOLCANIC	CP100007	CP100007
4" PLASTIC W/ POLY	CP100007	CP100007
4" VOLCANIC	CP100007	CP100007
4" PERFORMANCE W/ POLY	CP105139	CP105140
4" PERFORMANCE W/ SOFT ROLL	CP105139	CP105140
4" 1.5" WIDE POLY	CP100007	CP100007
5" PLASTIC W/ POLY	HD100084	HD100084
5" PERFORMANCE W/ POLY	CP105139	CP105140
5" PERFORMANCE W/ SOFT ROLL	CP105139	CP105140
5" 1.5" WIDE POLY	HD100084	HD100084
6" PLASTIC W/ POLY	HD100084	HD100084
6" PLASTIC W/ PNEUMATIC	HD100084	HD100084
6" PERFORMANCE W/ POLY	CP105139	CP105140
6" PERFORMANCE W/ PNEUMATIC	CP105139	CP105140
6" BILLET W/ SOFT ROLL	HD100084	HD100084
8" PLASTIC W/ PNEUMATIC	HD100084	HD100084

* SEE WHEEL ASSEMBLIES AND FORK ASSEMBLIES FOR INDIVIDUAL COMPONENTS


TiLite™ SX, YGS

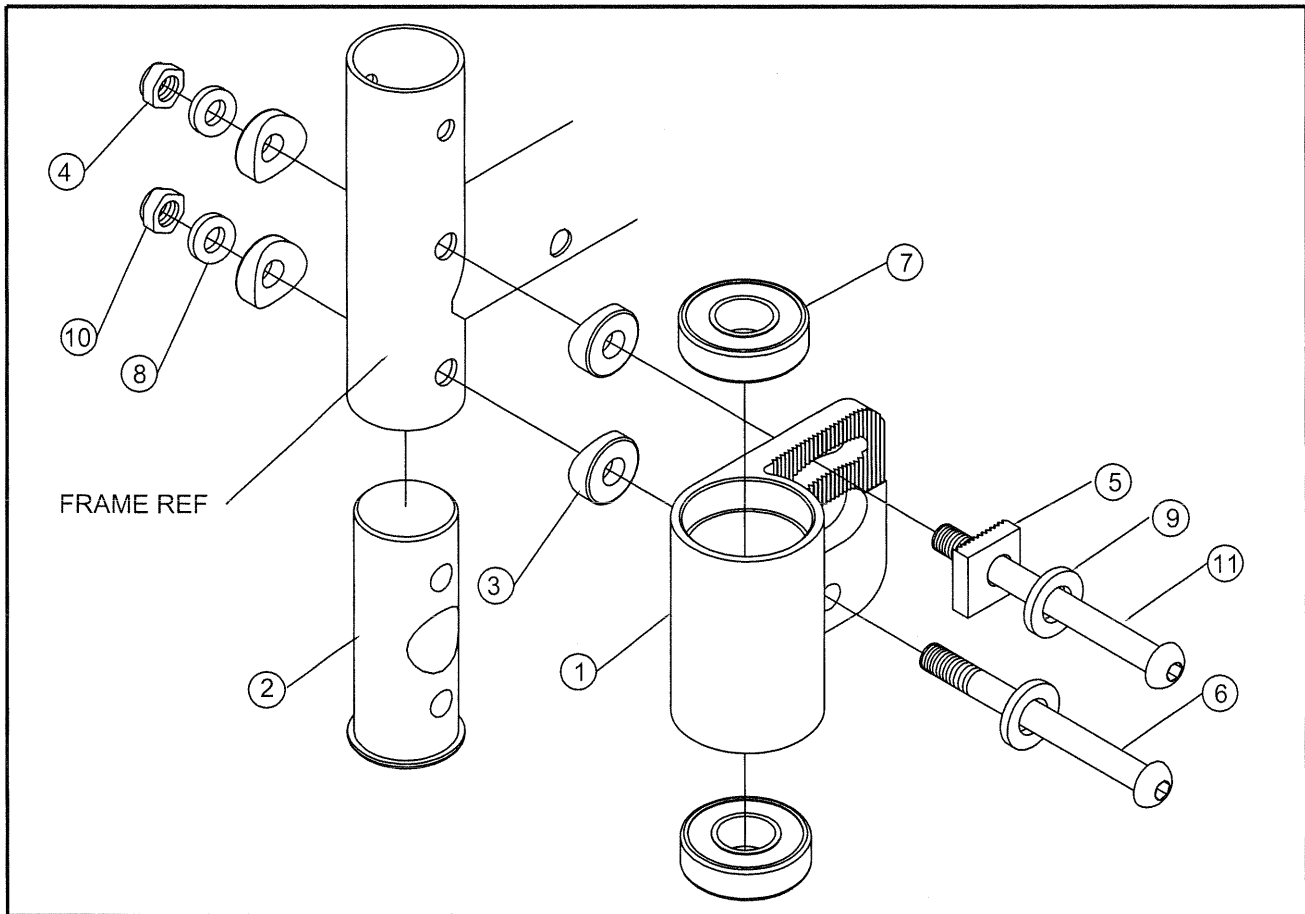

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100521	CASTER HOUSING ADJUSTABLE RIGHT
1B	1	CP100534	CASTER HOUSING ADJUSTABLE LEFT
2	1	CP100972	BOTTOM PLUG SWING AWAY
3	4	HD100240	CONCAVE WASHER
4	1	HD100021	1/4-28 X .218 TALL NYLOCK LOCKNUT
5	1	CP100853	LOCKING BLOCK FOR CASTER HOUSING
6	1	HD100104	1/4-28 X 2" BUTTON HEAD CAP SCREW
7	2	HD100011	BEARING 1/2 I.D. X 1-1/8 O.D. X 5/16 W
8	2	HD100022	1/4 SAE FLAT WASHER
9	2	HD100056	1/4" AN WASHER, .260 O.D. X .0625 THICK
10	1	HD100111	1/4-28 NYLOCK NUT
11	1	HD100242	1/4-28 X 2-1/8" BUTTON HEAD CAP SCREW
1A, 2-11		AS100642	CASTER HOUSING RIGHT ASSY
1B, 2-11		AS100643	CASTER HOUSING LEFT ASSY
1-11		AS100565	CASTER HOUSING ASSY PAIR


CASTER BARREL ASSEMBLY

TiLite™ X, SX , TRA, EVO, YRA, YG, YGS

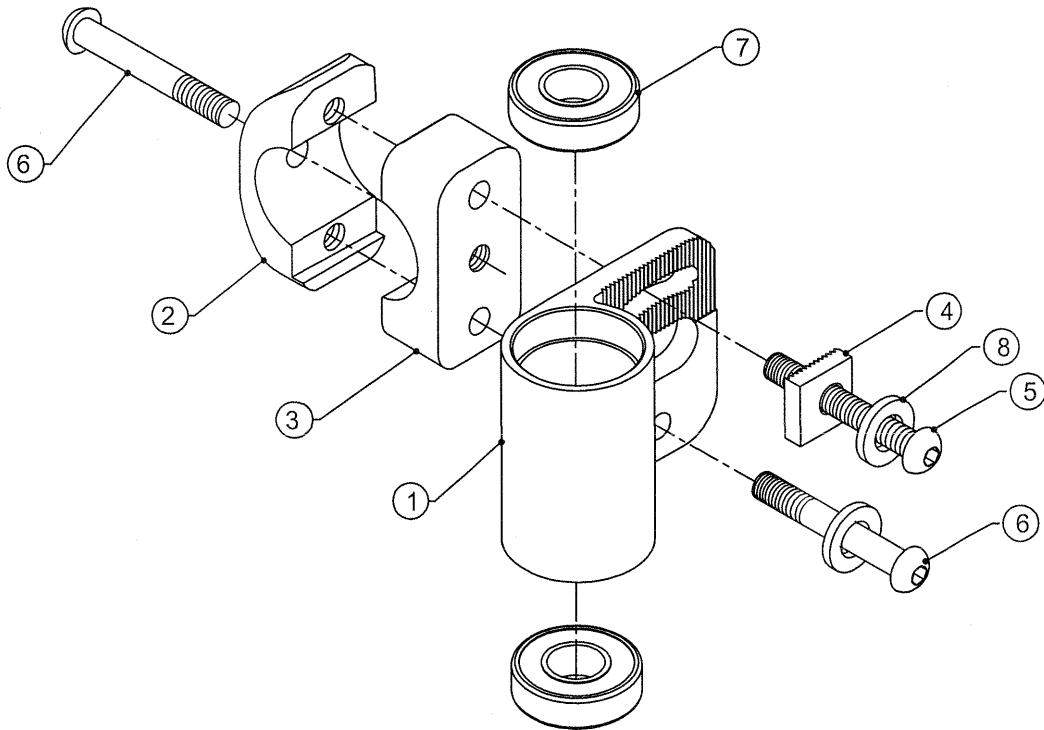

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100521	CASTER HOUSING ADJUSTABLE RIGHT
1B	1	CP100534	CASTER HOUSING ADJUSTABLE LEFT
2	2	CP100519	C CLAMP CASTER
3	2	CP100520	U CLAMP CASTER
4	2	CP100853	LOCKING BLOCK FOR CASTER HOUSING
5	2	HD100125	1/4-20 X 1-3/4 BUTTON HEAD CAP SCREW
6	4	HD100123	1/4-20 X 1-1/2 BUTTON HEAD CAP SCREW
7	4	HD100011	BEARING 1/2 I.D. X 1-1/8 O.D. X 5/16 W
8	2	HD100056	1/4" AN WASHER, .260 O.D. X .0625 THICK
1-8		AS100274	CASTER HOUSING ASSEMBLY PAIR
1A,7		AS101966	CASTER HOUSING STOCKED - RIGHT
1B,7		AS101967	CASTER HOUSING STOCKED - LEFT


3" CASTER WHEEL

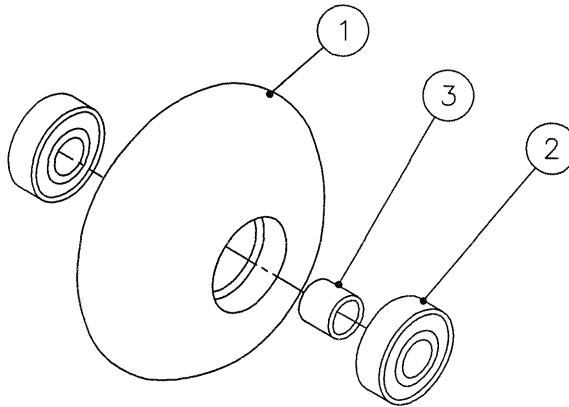

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100199	HYPER X-360- 82mm, 82A
2	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W
3	1	CP100006	CASTER SPACER 3" AND 4"
1-3		AS100356	MICRO CASTER WHEEL ASSEMBLY

CASTER WHEEL


3" & 4" LIGHT UP

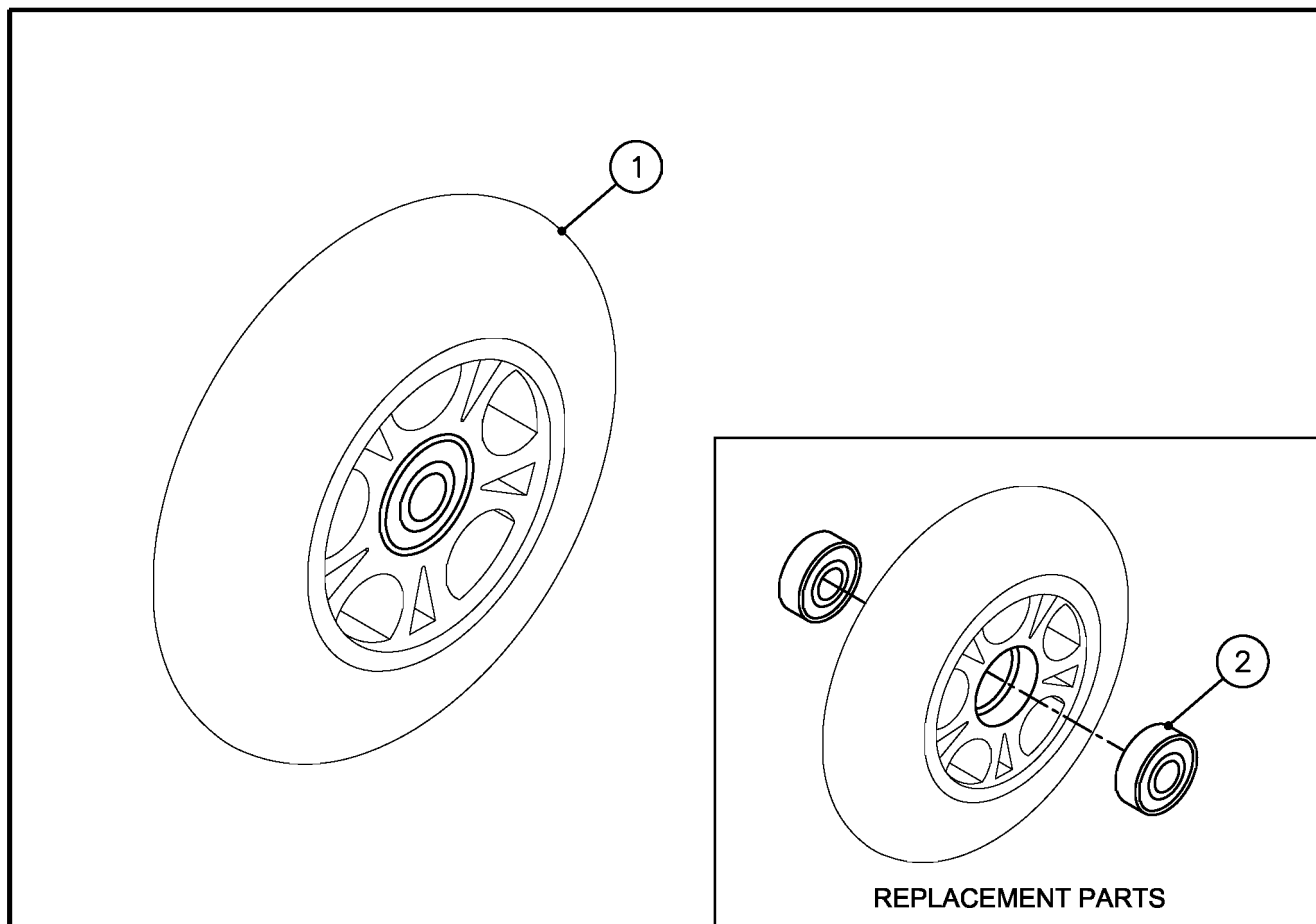

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP102034	4" VOLCANIC LIGHT UP WHEEL
1B	1	CP101436	3" VOLCANIC LIGHT UP WHEEL
2	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W

CASTER WHEEL

3" X 1.5" BILLET ALUMINUM MICROCASTER W/ SOFT ROLL TIRE FOR MONO FORK
TiLite™ TWIST, ZR (SERIES 2), ZRA (SERIES 2), AERO Z (SERIES 2),
TR (SERIES 3), AERO T

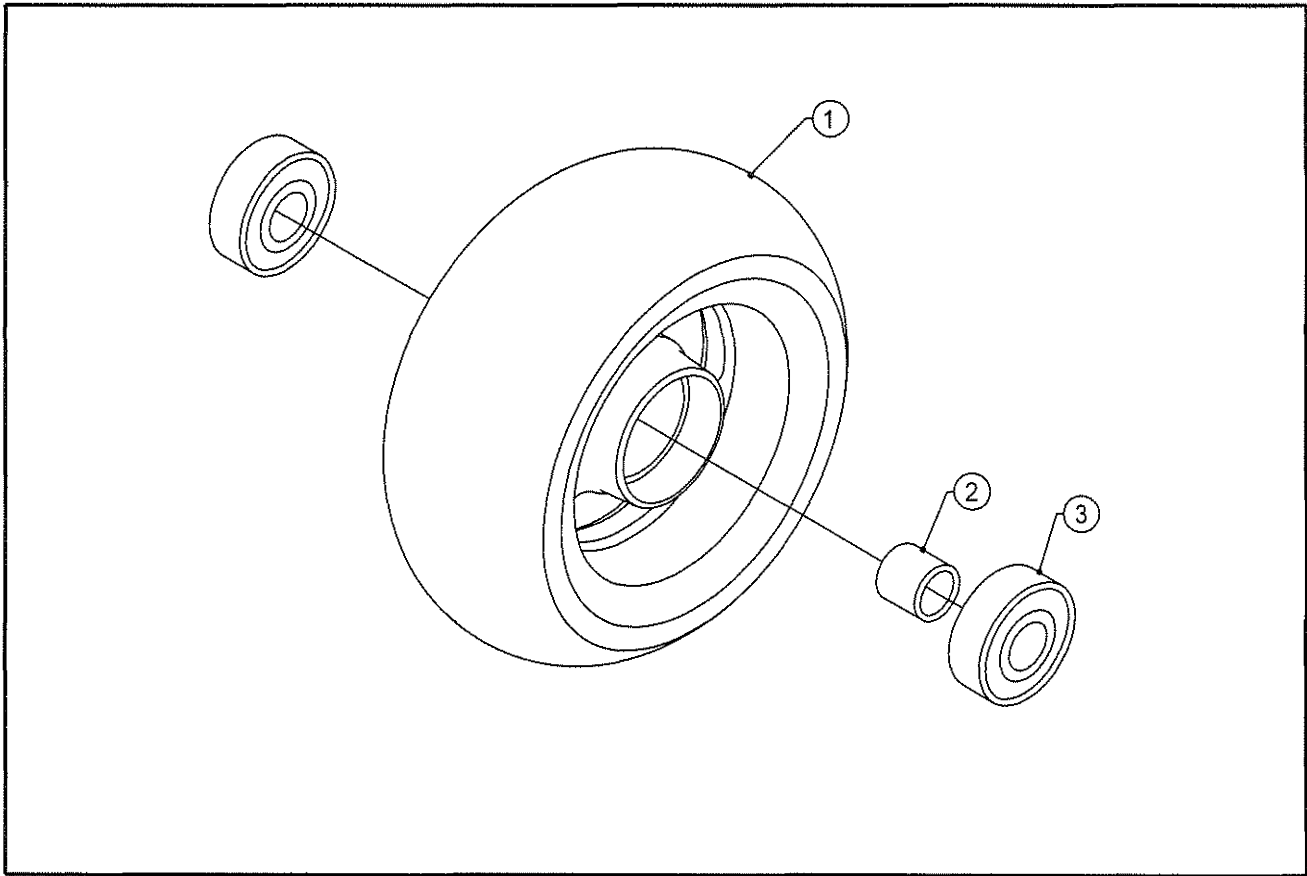


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP105133	3" FROGLEG SOFTROLL CASTER WITHOUT BEARINGS
2	1	CP104718	FROG LEG HUB SPACER
3	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED
1-3	1	AS111347	3" FROGLEG SOFTROLL CASTER

DISCONTINUED
SEE PAGE 8-9.3.2

CASTER WHEEL

1.5" WIDE 4" & 5" CASTER

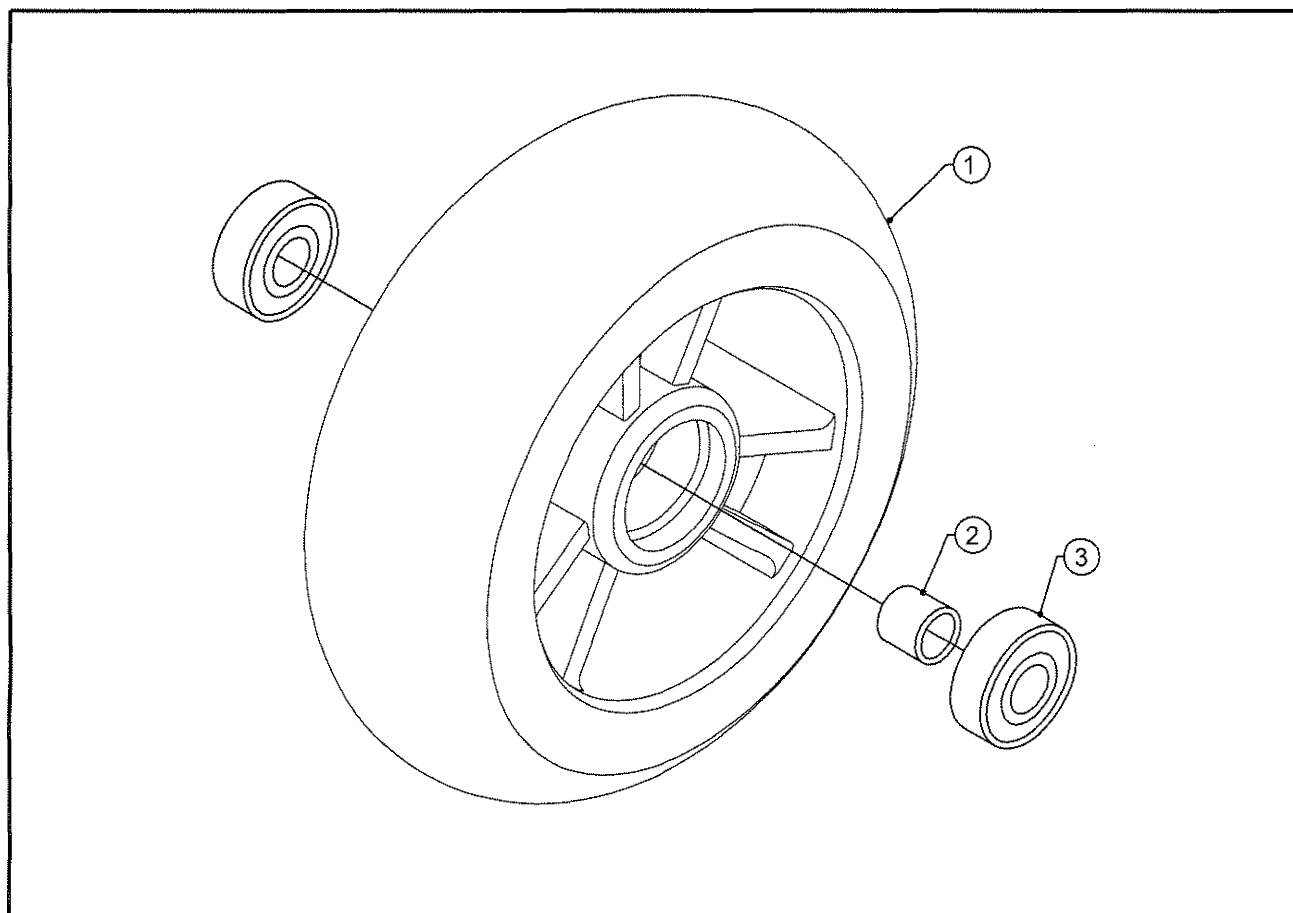


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A,2A,3A		AS104338	1.5" WIDE 4" POLY CASTER
1B,2A,3A		AS104339	1.5" WIDE 5" POLY CASTER
1A,2B,3B		AS110895	1.5" WIDE 4" POLY CASTER FOR MONO FORK
1B,2B,3B		AS110896	1.5" WIDE 5" POLY CASTER FOR MONO FORK
2A	1	HD100083	.500 X .450 SPACER
2B	1	CP104718	FROG LEG HUB SPACER (FOR MONO FORK)
3A	2	HD100012	BEARING 8mm I.D. X 22mm O.D. 7mm W
3B	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED (FOR MONO FORK)

DISCONTINUED
SEE PAGE 8-9.3.3

CASTER WHEEL

4" & 5" SOFT ROLL ASSY

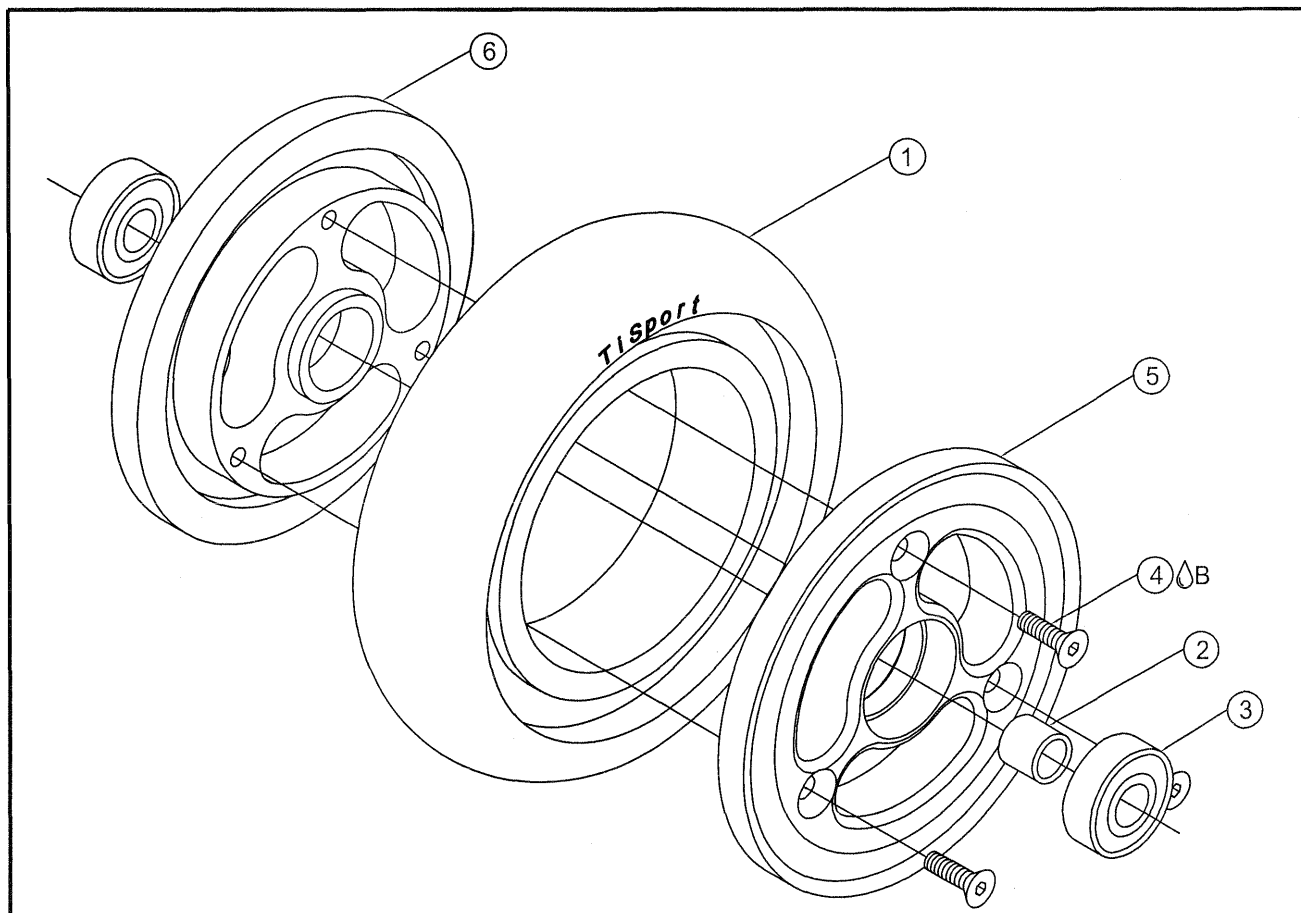


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP101303	4" SOFT ROLL TIRE
1B	1	CP102144	5" SOFT ROLL TIRE
2	1	CP100006	.438 X .400 SPACER
3	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W
4 ^B	3	HD100513	8-32 X 5/8" FLAT HEAD SOCKET CAP SCREW, BLACK ZINC
5A	1	CP101301	4" SOFT ROLL CASTER HUB A
5B	1	CP103558	4" BLACK ANODIZED SOFT ROLL CASTER HUB A
6A	1	CP101302	4" SOFT ROLL CASTER HUB B
6B	1	CP103559	4" BLACK ANODIZED SOFT ROLL CASTER HUB B
1A-4,5A,6A		AS101506	4" SOFT ROLL CASTER ASSY
1B-4,5A,6A		AS102748	5" SOFT ROLL CASTER ASSY
1A-4,5B,6B		AS103791	4" BLACK ANODIZED SOFT ROLL CASTER ASSY
1B-4,5B,6B		AS103792	5" BLACK ANODIZED SOFT ROLL CASTER ASSY

DISCONTINUED:
SEE PAGE 8-9.3.1

PERFORMANCE CASTER WHEEL

4" & 5" PERFORMANCE SOFT ROLL ASSY

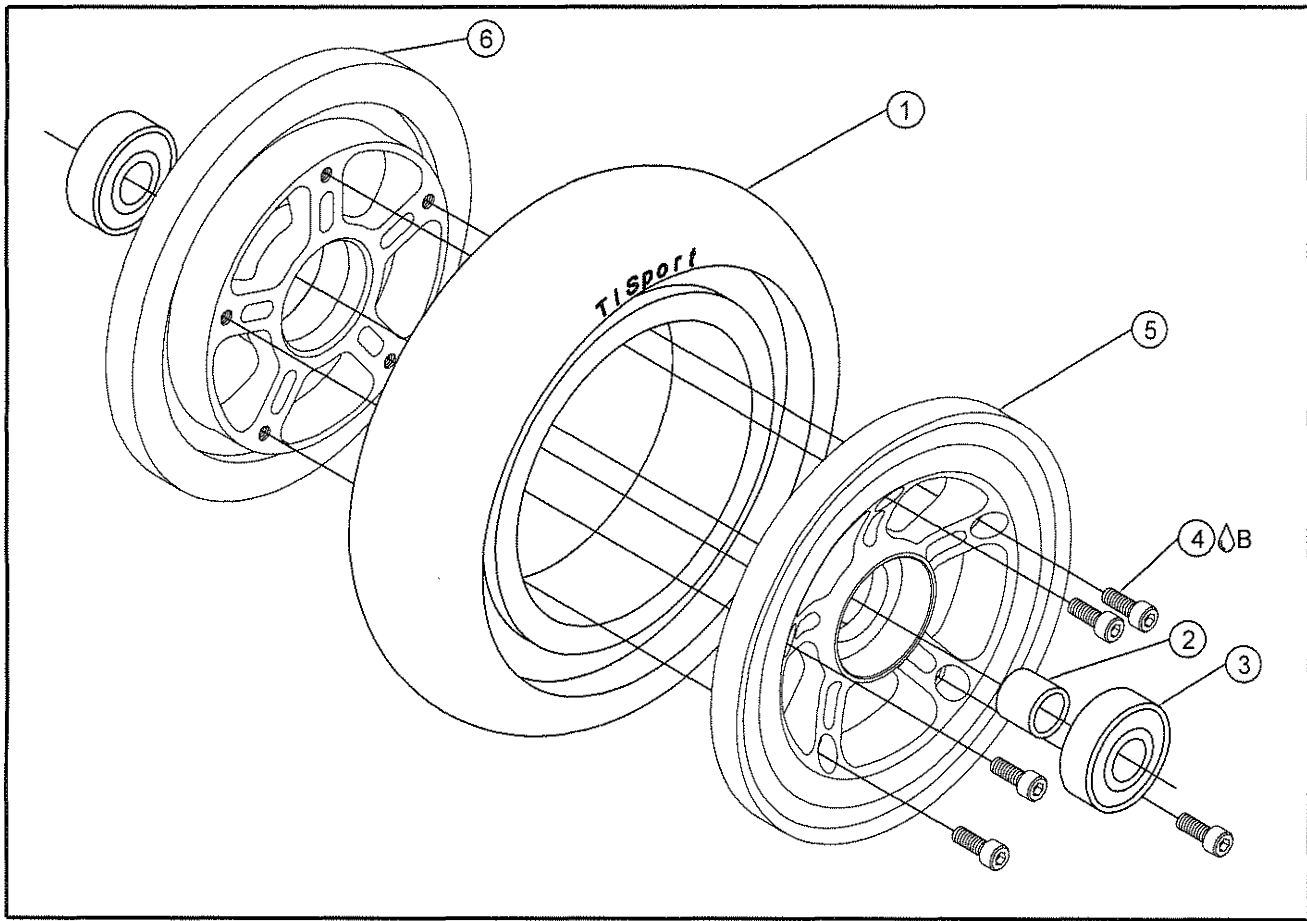

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP101303	4" SOFT ROLL TIRE
1B	1	CP102144	5" SOFT ROLL TIRE
2	1	CP104326	.384 ID X .500 OD X .231 SPACER
3	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED
4A/B	5	HD100620	6-32 X 3/8" SOCKET HEAD CAP SCREW, CLEAR ZINC
4B/B	5	HD100419	6-32 X 3/8" SOCKET HEAD CAP SCREW, BLACK
5A	1	CP104300	PERFORMANCE SOFT ROLL HUB A
5B	1	CP104367	PERFORMANCE SOFT ROLL HUB A, BLACK
6A	1	CP104301	PERFORMANCE SOFT ROLL HUB B
6B	1	CP104368	PERFORMANCE SOFT ROLL HUB B, BLACK
1A-3,4A,5A,6A		AS110897	4" PERFORMANCE SOFT ROLL CASTER ASSY
1B-3,4A,5A,6A		AS110898	5" PERFORMANCE SOFT ROLL CASTER ASSY
1A-3,4B,5B,6B		AS110899	4" PERFORMANCE SOFT ROLL CASTER ASSY, BLACK
1B-3,4B,5B,6B		AS110900	5" PERFORMANCE SOFT ROLL CASTER ASSY, BLACK


DISCONTINUED



Parts Manual

LITESPEED SOFT ROLL CASTER WHEEL


3", 4", 5", & 6" LITESPEED SOFT ROLL CASTER WHEELS W/ TIRE

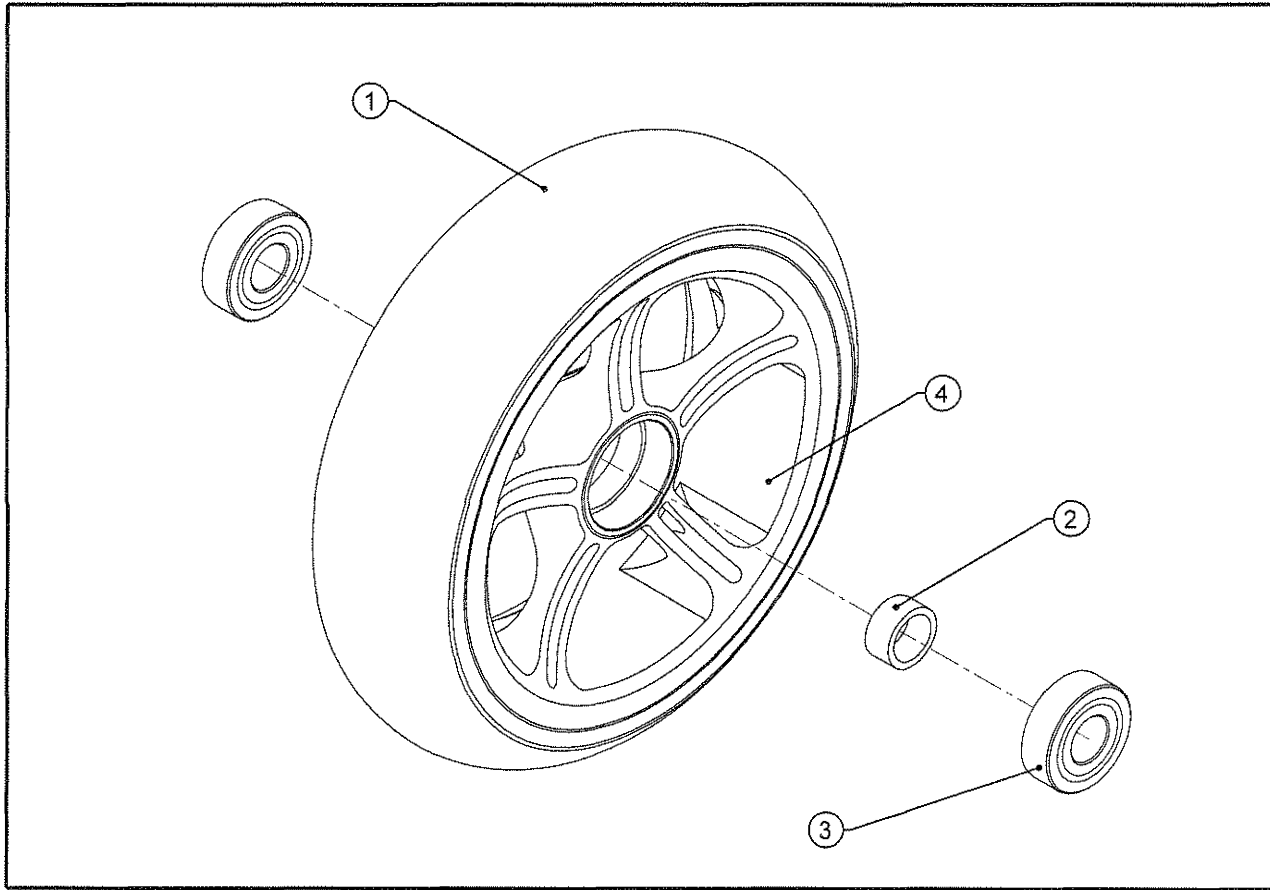

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP105875	3 INCH SOFT ROLL TIRE FOR 1.5" WIDE CASTER
1B	1	CP105876	4 INCH SOFT ROLL TIRE FOR 1.5" WIDE CASTER
1C	1	CP105877	5 INCH SOFT ROLL TIRE FOR 1.5" WIDE CASTER
1D	1	CP105878	6 INCH SOFT ROLL TIRE FOR 1.5" WIDE CASTER
2	1	CP104326	.384 ID X .500 OD X .231 SPACER
3	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED
4A	1	CP105916	3" X 1.5" HUB
4B	1	CP105917	4" X 1.5" HUB
4C	1	CP105918	5" X 1.5" HUB
4D	1	CP105919	6" X 1.5" HUB
4E	1	CP106229	3" X 1.5" HUB, BLACK
4F	1	CP106230	4" X 1.5" HUB, BLACK
4G	1	CP106231	5" X 1.5" HUB, BLACK
4H	1	CP106232	6" X 1.5" HUB, BLACK
1A,2,3,4A		AS112008	3" X 1.5" WIDE SOFT ROLL CASTER
1B,2,3,4B		AS112009	4" X 1.5" WIDE SOFT ROLL CASTER
1C,2,3,4C		AS112010	5" X 1.5" WIDE SOFT ROLL CASTER
1D,2,3,4D		AS112011	6" X 1.5" WIDE SOFT ROLL CASTER
1A,2,3,4E		AS112012	3" X 1.5" WIDE SOFT ROLL CASTER, BLACK
1B,2,3,4F		AS112013	4" X 1.5" WIDE SOFT ROLL CASTER, BLACK
1C,2,3,4G		AS112014	5" X 1.5" WIDE SOFT ROLL CASTER, BLACK
1D,2,3,4H		AS112015	6" X 1.5" WIDE SOFT ROLL CASTER, BLACK

PLASTIC SOFT ROLL CASTER WHEEL


4", 5", & 6" LITESPEED PLASTIC W/ SOFT ROLL CASTER WHEELS

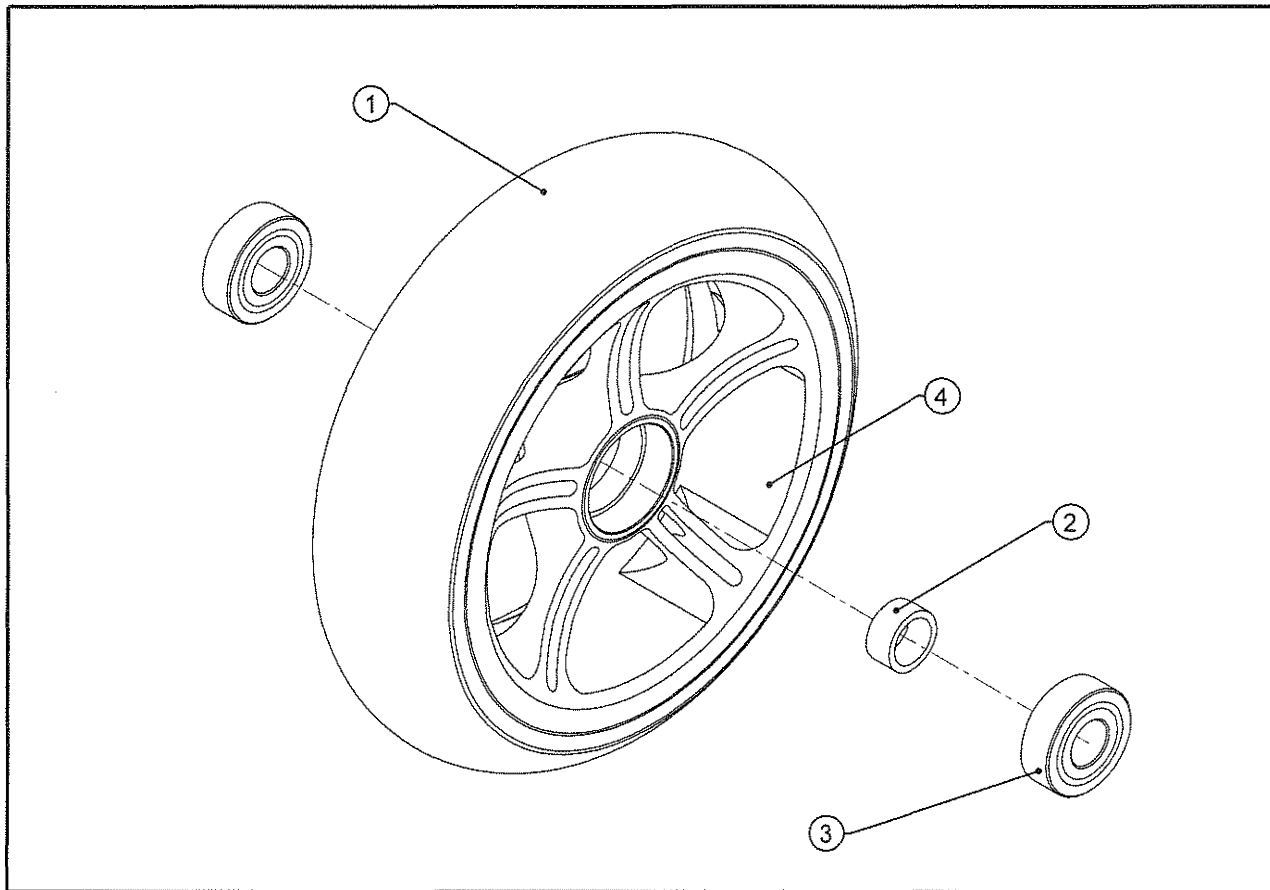

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP105876	4 INCH SOFT ROLL TIRE FOR 1.5" WIDE CASTER
1B	1	CP105877	5 INCH SOFT ROLL TIRE FOR 1.5" WIDE CASTER
1C	1	CP105878	6 INCH SOFT ROLL TIRE FOR 1.5" WIDE CASTER
2	1	CP104326	.384 ID X .500 OD X .231 SPACER
3	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED
4A	1	CP106264	4" X 1.5" COMPOSITE HUB
4B	1	CP106265	5" X 1.5" COMPOSITE HUB
4C	1	CP106266	6" X 1.5" COMPOSITE HUB
1A,2,3,4A		AS112041	4" X 1.5" COMPOSITE WIDE SOFT ROLL CASTER
1B,2,3,4B		AS112042	5" X 1.5" COMPOSITE WIDE SOFT ROLL CASTER
1C,2,3,4C		AS112043	6" X 1.5" COMPOSITE WIDE SOFT ROLL CASTER

CASTER WHEEL

6" SOFT ROLL ASSY

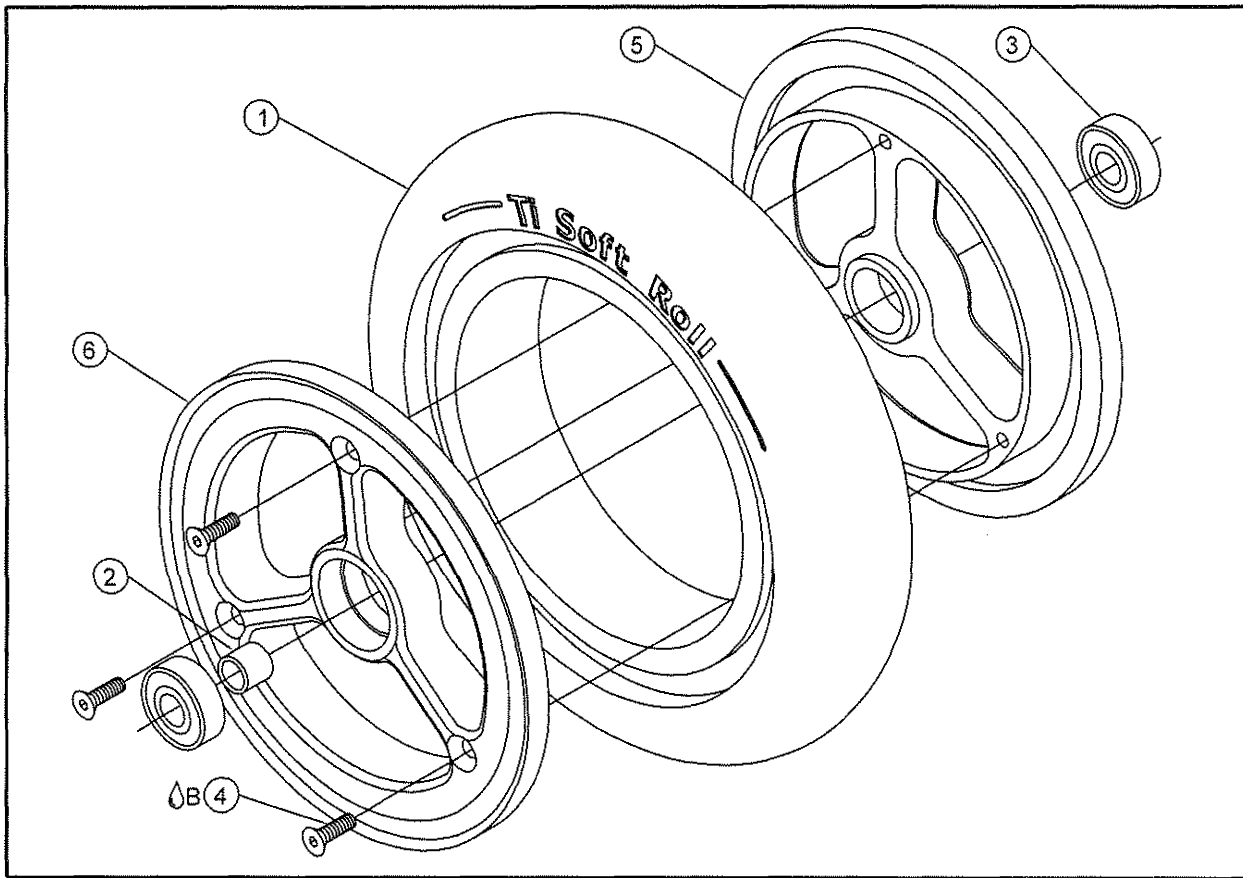


ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP101491	6" SOFT ROLL TIRE
2	1	HD100083	.500 X .450 SPACER
3	2	HD100012	BEARING 8mm I.D. X 22mm O.D. 7mm W
4 Δ B	3	HD100299	8-32 X 7/8" FLAT HEAD SOCKET CAP SCREW
5	1	CP101492	6" SOFT ROLL CASTER HUB A
6	1	CP101493	6" SOFT ROLL CASTER HUB B
1-6		AS101798	6" SOFT ROLL CASTER ASSY

DISCONTINUED

CASTER WHEEL

4", 5", & 6" ALUMINUM CASTER WHEELS/URETHANE TIRES

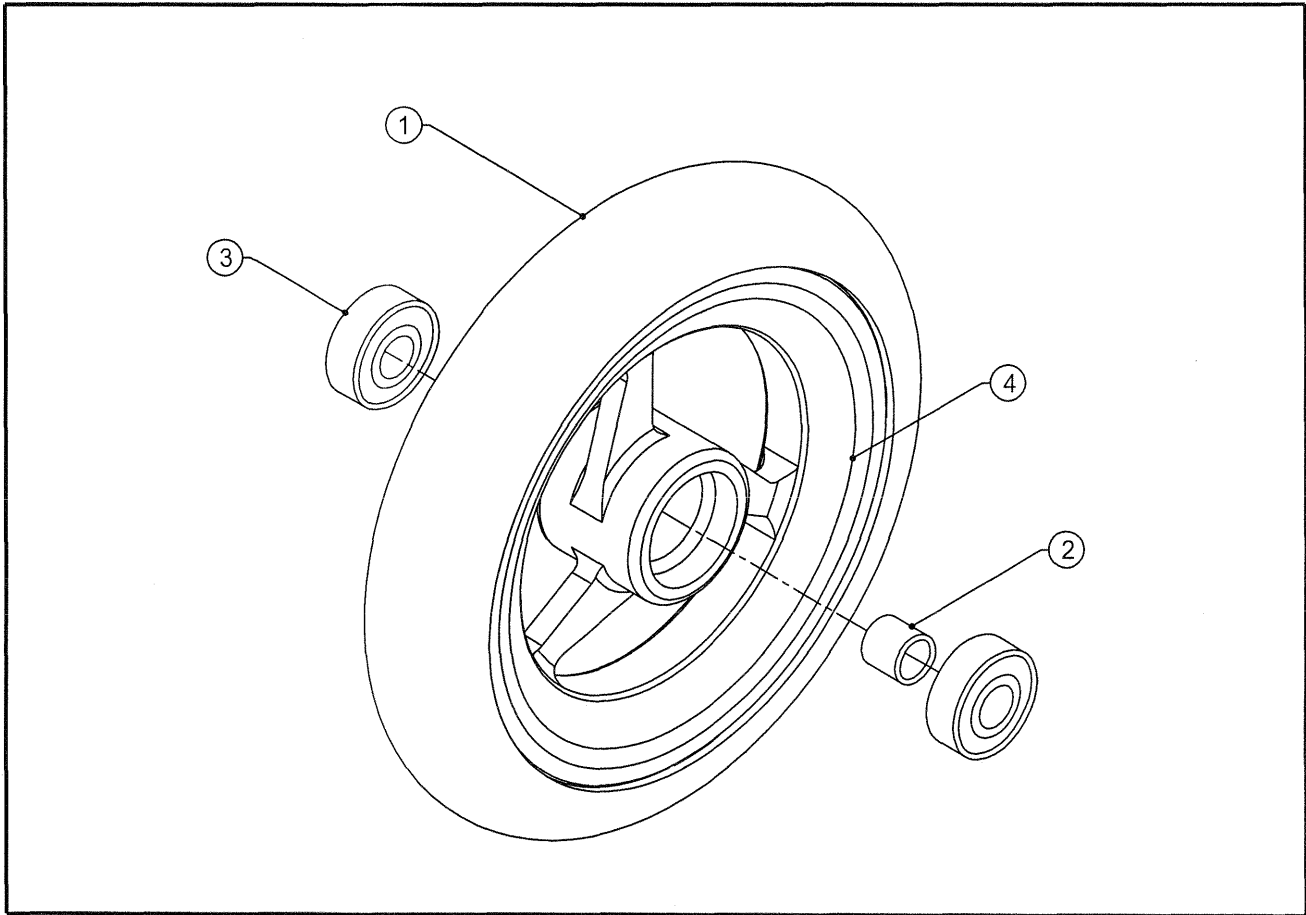


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100759	4" HIGH RESILIENT BLACK TIRE
1B	1	CP100340	5" POLYURETHANE TIRE
1C	1	CP100341	6" POLYURETHANE TIRE
2A	1	CP100006	.438 X .400 SPACER
2B	1	HD100083	.438 X .450 SPACER
3	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W
4A	1	CP100008	4" ALUMINUM WHEEL
4B	1	CP100358	5" ALUMINUM WHEEL
4C	1	CP103556	4" ALUMINUM BLACK ANODIZED WHEEL
4D	1	CP103557	5" ALUMINUM BLACK ANODIZED WHEEL
1A,2A,3,4A		AS100002	4" ALUMINUM POLY CASTER
1B,2B,3,4B		AS100073	5" ALUMINUM POLY CASTER
1C,2B,3,4B		AS100074	6" ALUMINUM POLY CASTER
1A,2A,3,4C		AS103787	4" ALUMINUM BLACK ANODIZED POLY CASTER
1B,2B,3,4D		AS103788	5" ALUMINUM BLACK ANODIZED POLY CASTER
1C,2B,3,4D		AS103789	6" ALUMINUM BLACK ANODIZED POLY CASTER

DISCONTINUED:
SEE PAGE 8-9.5.1

LITESPEED CASTER WHEEL


4", 5", & 6" LITESPEED CASTER WHEELS/URETHANE TIRES

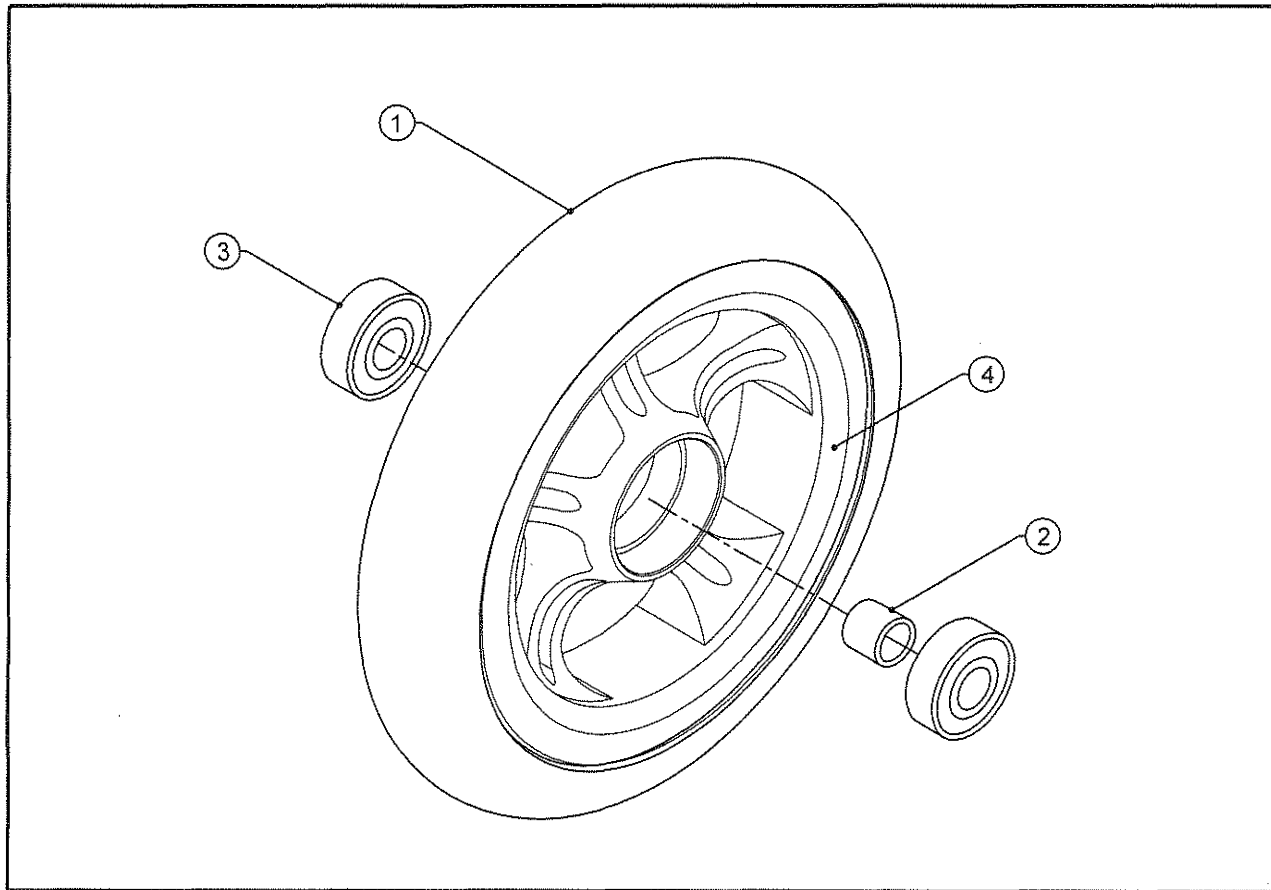

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100759	4" POLYURETHANE TIRE
1B	1	CP100340	5" POLYURETHANE TIRE
1C	1	CP100341	6" POLYURETHANE TIRE
2	1	CP104326	.384 ID X .500 OD X .231 SPACER
3	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED
4A	1	CP104388	4" PERFORMANCE POLY HUB
4B	1	CP104389	5"-6" PERFORMANCE POLY HUB
4C	1	CP104366	4" PERFORMANCE POLY HUB, BLACK
4D	1	CP104369	5"-6" PERFORMANCE POLY HUB, BLACK
1A,2,3,4A		AS110901	4" PERFORMANCE POLY CASTER
1B,2,3,4B		AS110902	5" PERFORMANCE POLY CASTER
1C,2,3,4B		AS110903	6" PERFORMANCE POLY CASTER
1A,2,3,4C		AS110904	4" PERFORMANCE POLY CASTER, BLACK
1B,2,3,4D		AS110905	5" PERFORMANCE POLY CASTER, BLACK
1C,2,3,4D		AS110906	6" PERFORMANCE POLY CASTER, BLACK

CASTER WHEEL


4" PLASTIC POLY CASTER

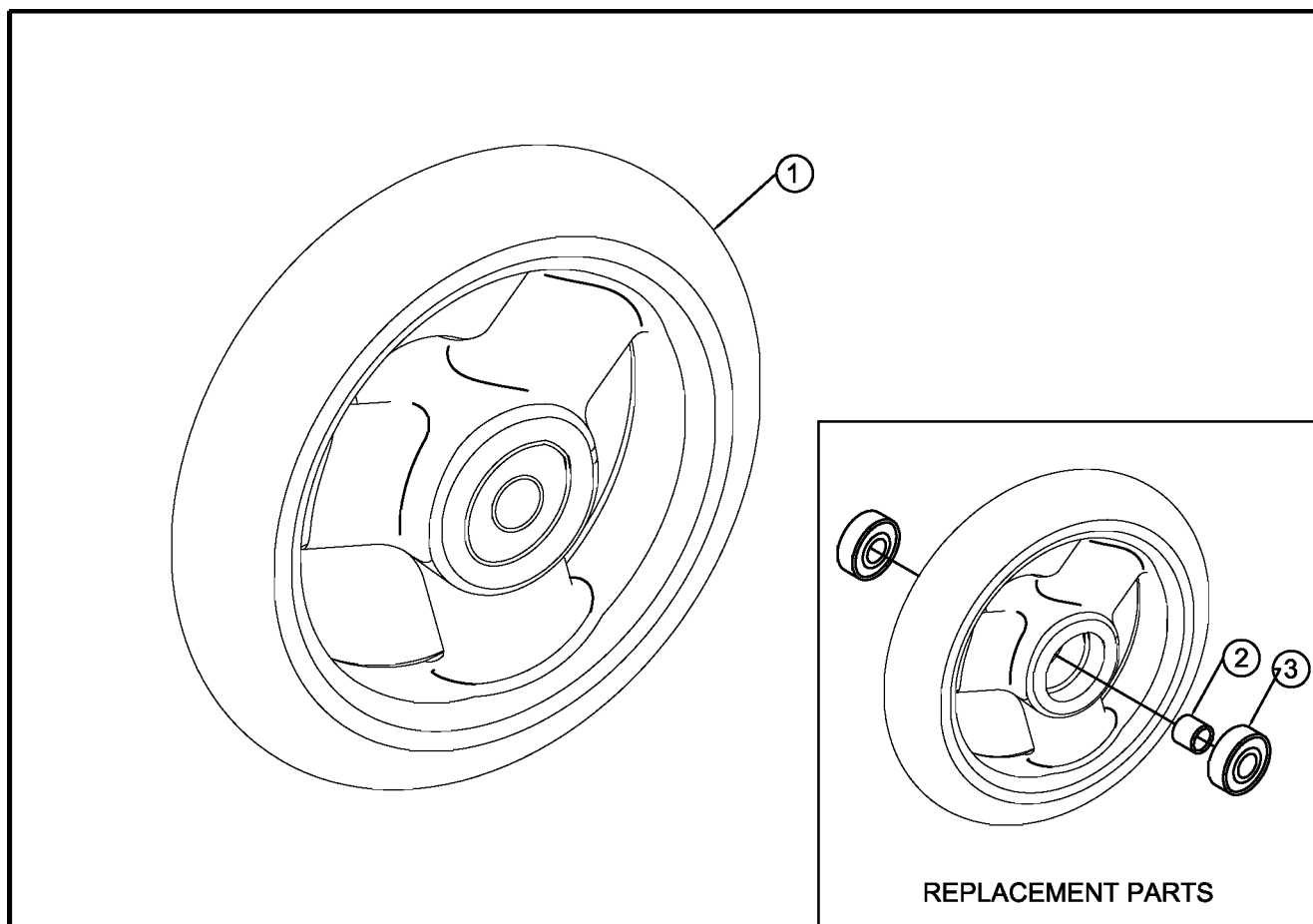

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP103491	4" PLASTIC POLY CASTER
2	1	HD100083	.500 X .450 SPACER
3	2	HD100012	BEARING 8mm I.D. X 22mm O.D. 7mm W

CASTER WHEEL


5", & 6" PLASTIC CASTER WHEELS/URETHANE TIRES

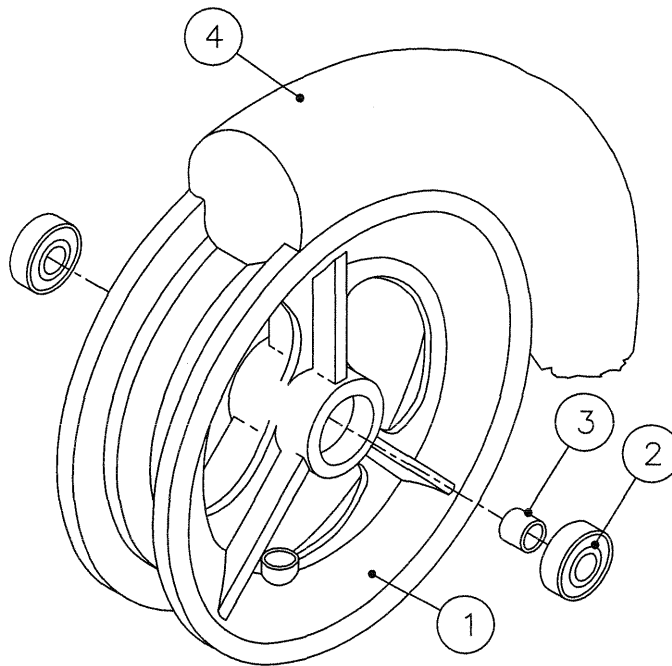

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100332	PLASTIC WHEEL
2	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W
3	1	HD100083	.438 X .450 SPACER
4A	1	CP100340	5" URETHANE TIRE
4B	1	CP100341	6" URETHANE TIRE
1-3, 4A		AS100064	5" PLASTIC POLY CASTER ASSY
1-3, 4B		AS100066	6" PLASTIC POLY CASTER ASSY

CASTER WHEEL


8" PLASTIC CASTER WHEELS/URETHANE TIRES

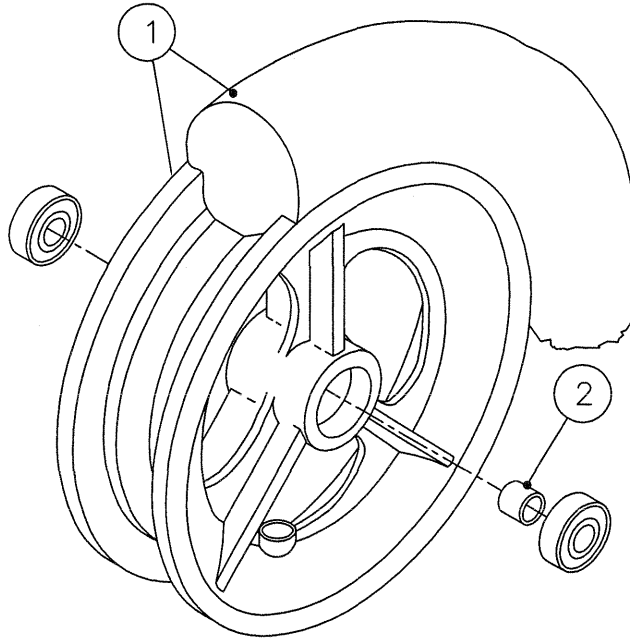

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100751	PLASTIC WHEEL/URETHANE TIRE
2	1	HD100083	.438 X .450 SPACER
1-2		AS100399	8" PLASTIC POLY CASTER ASSY

CASTER WHEEL


6" PLASTIC CASTER WHEEL/PNEUMATIC TIRES

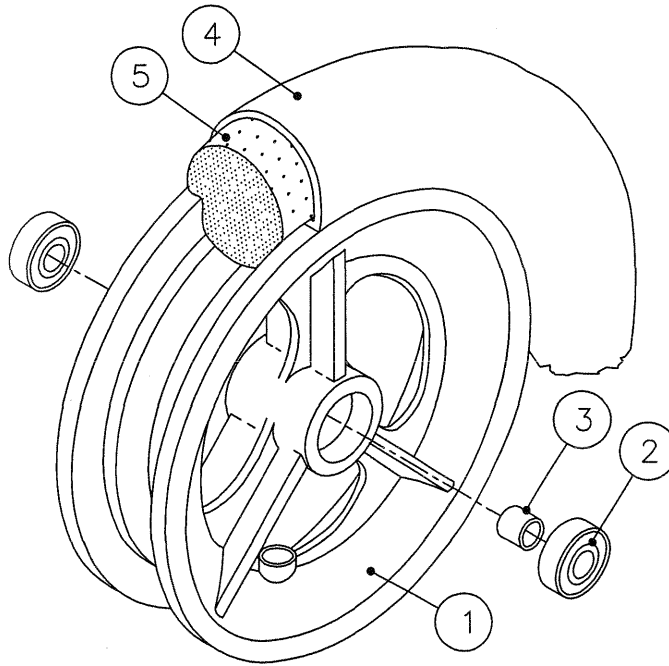

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100332	PLASTIC WHEEL
2	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W
3	1	HD100083	.438 X .450 SPACER
4	1	CP100333	6" PNEUMATIC TIRE
5	1	CP100334	6" TUBE WITH BENT STEM
1-5		AS100068	6" PLASTIC PNEUMATIC CASTER ASSY

CASTER WHEEL

6" ALUMINUM CASTER WHEELS/PNEUMATIC TIRES

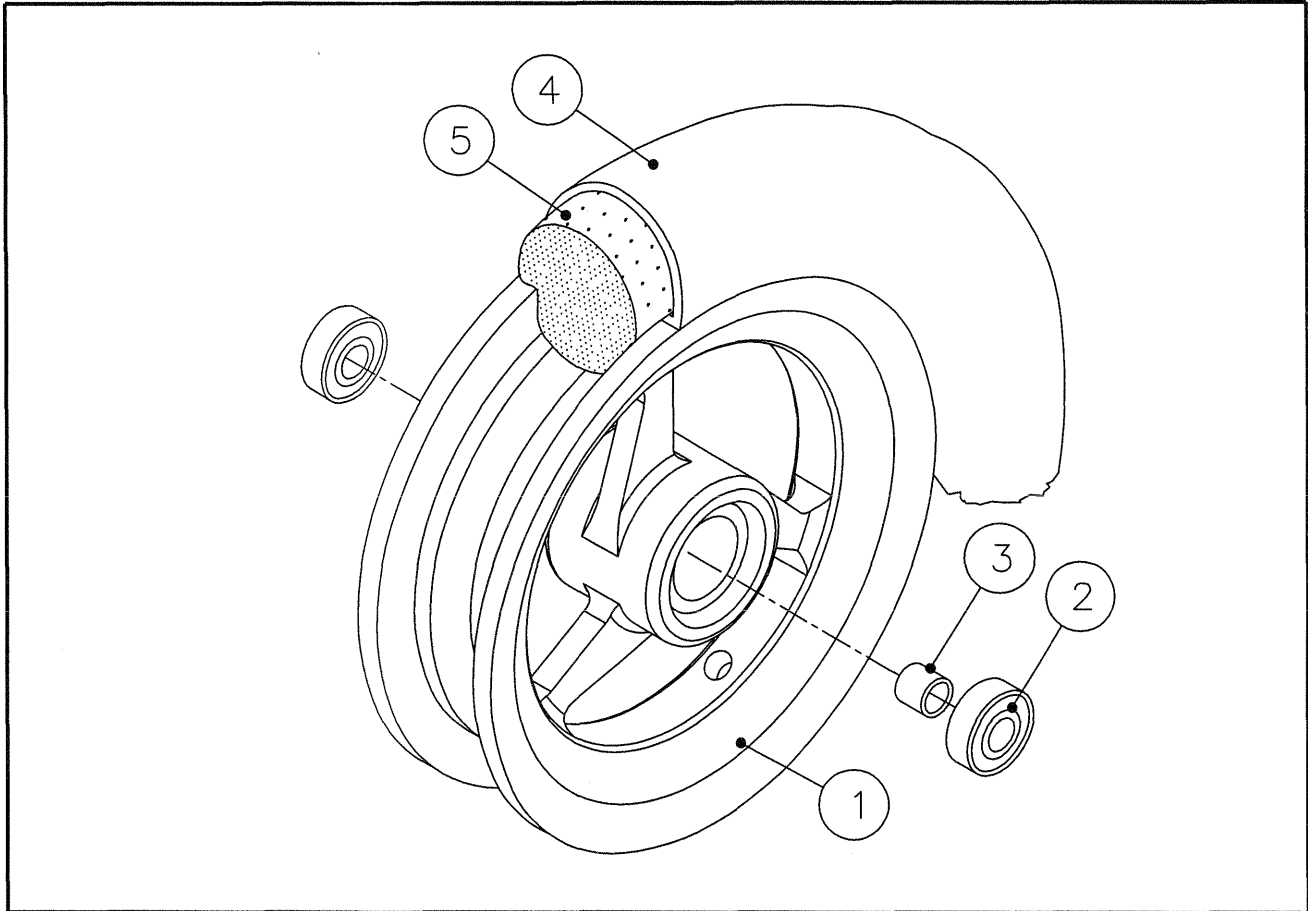


ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP100358	5" ALUMINUM WHEEL
1B	1	CP103557	5" ALUMINUM BLACK ANODIZED WHEEL
2	2	HD100012	BEARING, 8mm I.D. X 22mm O.D. X 7mm W
3	1	HD100083	.438 X .450 SPACER
4	1	CP100333	6" PNEUMATIC TIRE
5	1	CP100334	6" TUBE WITH BENT STEM
1A-5		AS100075	6" ALUMINUM PNEUMATIC CASTER
1B-5		AS103790	6" ALUMINUM BLACK ANODIZED PNEUMATIC CASTER

DISCONTINUED:
SEE PAGE 8-9.10.1

LITESPEED CASTER WHEEL


6" LITESPEED CASTER WHEELS/PNEUMATIC TIRES

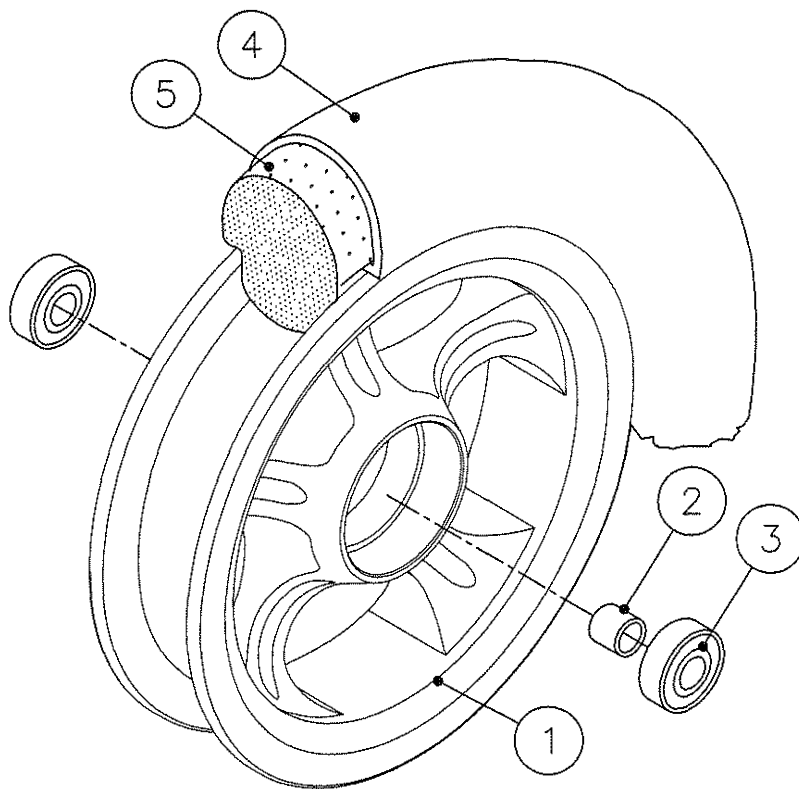

ITEM	QUANTITY	PART NO.	DESCRIPTION
1A	1	CP104389	5"-6" PERFORMANCE POLY HUB
1B	1	CP104369	5"-6" PERFORMANCE POLY HUB, BLACK
2	1	CP104326	.384 ID X .500 OD X .231 SPACER
3	2	HD100516	BEARING, R6, 7/8" OD, 3/8" ID, 9/32" WIDE, DOUBLE SEALED
4	1	CP100333	6" PNEUMATIC TIRE
5	1	CP100334	6" TUBE WITH BENT STEM
1A-5		AS110907	6" PERFORMANCE PNEUMATIC CASTER
1B-5		AS110908	6" PERFORMANCE PNEUMATIC CASTER, BLACK

CASTER WHEEL

8" PLASTIC CASTER WHEEL/PNEUMATIC TIRES

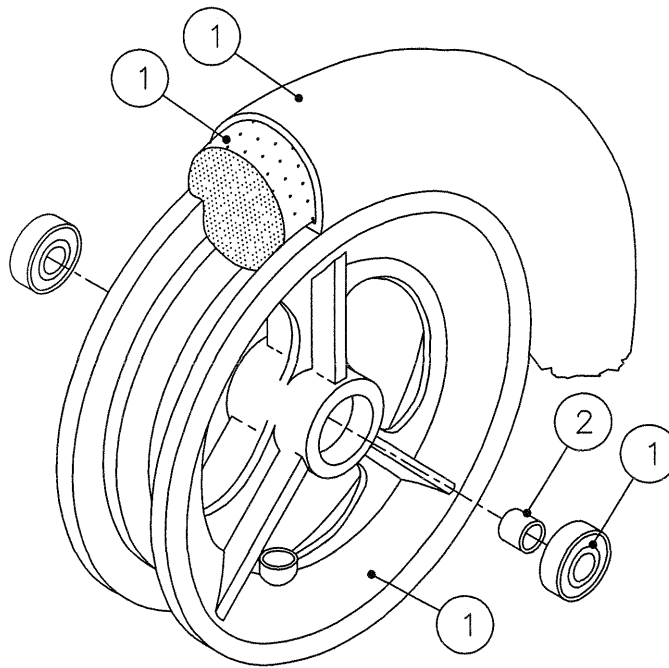

ITEM	QUANTITY	PART NO.	DESCRIPTION
1	1	CP100752	PLASTIC WHEEL WITH PNEUMATIC TIRE
2	1	HD100083	.438 X .450 SPACER
1-2		AS100400	8" PLASTIC PNEUMATIC CASTER

CASTER COMBINATIONS 2002

PART NO.	DESCRIPTION
TiLite™ TR/TRC/XC	
AS101119	3" WHEEL ASSY 2002
AS101472	3" LIGHTED WHEEL ASSY 2002
AS102591	4" LIGHTED WHEEL ASSY 2003
AS103876	4" PLASTIC POLY WHEEL ASSY
AS101120	4" ALUMINUM WHEEL ASSY 2002
AS103906	4" BLACK ALUMINUM WHEEL ASSY
AS101473	4" SOFT ROLL WHEEL ASSY 2002
AS103956	4" SOFT ROLL BLACK WHEEL ASSY
AS101121	3" WHEEL TI STEM ASSY 2002
AS101474	3" LIGHTED WHEEL TI STEM ASSY 2002
AS102592	4" LIGHTED WHEEL TI STEM ASSY 2003
AS101122	4" ALUMINUM WHEEL TI STEM ASSY 2002
AS101475	4" SOFT ROLL WHEEL TI STEM ASSY 2002
AS101123	5" POLY W/ PLASTIC WHEEL 5" SPECIAL FORK ASSY 2002
AS101124	5" POLY W/ ALUMINUM WHEEL 5" SPECIAL FORK ASSY 2002
AS103920	5" POLY W/ BLACK ALUMINUM WHEEL 5" SPECIAL FORK ASSY
AS102749	5" SOFT ROLL WHEEL ASSY 5" SPECIAL FORK ASSY
AS103970	5" SOFT ROLL BLACK WHEEL ASSY 5" SPECIAL FORK ASSY
AS101125	5" POLY W/ PLASTIC WHEEL TI STEM ASSY 2002
AS101126	5" POLY W/ ALUMINUM WHEEL TI STEM ASSY 2002
AS102750	5" SOFT ROLL WHEEL ASSY 5" SPECIAL FORK ASSY
TiLite™ XC	
AS101127	6" POLY W/ ALUMINUM WHEEL ASSY 2002
AS103936	6" POLY W/ BLACK ALUMINUM WHEEL ASSY
AS101128	6" POLY W/ PLASTIC WHEEL ASSY 2002
AS101129	6" PNEUMATIC W/ PLASTIC WHEEL ASSY 2002
AS101130	6" PNEUMATIC W/ ALUMINUM WHEEL ASSY 2002
AS103946	6" PNEUMATIC W/ BLACK ALUMINUM WHEEL ASSY
AS101799	6" SOFT ROLL WHEEL ASSY 2002
AS101800	6" SOFT ROLL WHEEL TI STEM ASSY 2002
TiLite™ X/SX	
AS101119	3" WHEEL ASSY 2002
AS101472	3" LIGHTED WHEEL ASSY 2002
AS102591	4" LIGHTED WHEEL ASSY 2003
AS101120	4" WHEEL W/ 5" & 6" FORK ASSY 2002
AS101473	3" LIGHTED WHEEL W/ 5" & 6" FORK ASSY 2002
AS102593	4" LIGHTED WHEEL W/ 5" & 6" FORK ASSY 2003
AS101474	3" WHEEL W/ 7" & 8" FORK ASSY 2002
AS102594	3" LIGHTED WHEEL W/ 7" & 8" FORK ASSY 2002
AS102594	4" LIGHTED WHEEL W/ 7" & 8" FORK ASSY 2003
AS103786	4" PLASTIC POLY WHEEL ASSY
AS101120	4" ALUMINUM WHEEL ASSY 2002
AS103906	4" BLACK ALUMINUM WHEEL ASSY
AS101473	4" SOFT ROLL WHEEL ASSY 2002
AS103956	4" SOFT ROLL BLACK WHEEL ASSY
AS103890	4" PLASTIC POLY WHEEL W/ 5" & 6" FORK ASSY
AS101133	4" WHEEL W/ 5" & 6" FORK ASSY 2002
AS101478	4" SOFT ROLL WHEEL W/ 5" & 6" FORK ASSY 2002
AS103891	4" PLASTIC POLY WHEEL W/ 7" & 8" FORK ASSY
AS101134	4" WHEEL W/ 7" & 8" FORK ASSY 2002
AS101479	4" SOFT ROLL WHEEL W/ 7" & 8" FORK ASSY 2002
AS103957	4" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK ASSY
AS101135	5" POLY W/ PLASTIC WHEEL ASSY 2002
AS101136	5" POLY W/ ALUMINUM WHEEL ASSY 2002
AS103921	5" POLY W/ BLACK ALUMINUM WHEEL ASSY
AS102751	5" SOFT ROLL WHEEL ASSY W/ 5" & 6" FORK ASSY
AS103971	5" SOFT ROLL BLACK WHEEL ASSY W/ 5" & 6" FORK ASSY
AS101127	6" POLY W/ ALUMINUM WHEEL ASSY 2002
AS103936	6" POLY W/ BLACK ALUMINUM WHEEL ASSY

SEE WHEEL ASSEMBLIES AND FORK ASSEMBLIES FOR INDIVIDUAL COMPONENTS



CASTER COMBINATIONS 2002 (CONT.)

PART NO.	DESCRIPTION
TiLite X/SX	
AS101128	6" POLY W/ PLASTIC WHEEL ASSY 2002
AS101129	6" PNEUMATIC W/ PLASTIC WHEEL ASSY 2002
AS101130	6" PNEUMATIC W/ ALUMINUM WHEEL ASSY 2002
AS103946	6" PNEUMATIC W/ BLACK ALUMINUM WHEEL ASSY
AS101799	6" SOFT ROLL WHEEL ASSY 2002
AS101137	5" POLY W/ PLASTIC WHEEL W/ 7" & 8" FORK ASSY 2002
AS101138	5" POLY W/ ALUMINUM WHEEL W/ 7" & 8" FORK ASSY 2002
AS103922	5" POLY W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK ASSY
AS102752	5" SOFT ROLL WHEEL ASSY W/ 7" & 8" FORK ASSY
AS103972	5" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK ASSY
AS101139	6" POLY W/ ALUMINUM WHEEL W/ 7" & 8" FORK ASSY 2002
AS103937	6" POLY W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK ASSY
AS101140	6" POLY W/ PLASTIC WHEEL W/ 7" & 8" FORK ASSY 2002
AS101141	6" PNEUMATIC W/ ALUMINUM WHEEL W/ 7" & 8" FORK ASSY 2002
AS103947	6" PNEUMATIC W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK ASSY
AS101142	6" PNEUMATIC W/ PLASTIC WHEEL W/ 7" & 8" FORK ASSY 2002
AS101801	6" SOFT ROLL WHEEL W/ 7" & 8" FORK ASSY 2002
AS101143	8" POLY W/ PLASTIC WHEEL W/ 7" & 8" FORK ASSY 2002
AS101144	8" PNEUMATIC W/ PLASTIC WHEEL W/ 7" & 8" FORK ASSY 2002
TiLite X/SX WITH 1/2" EXTENSION	
AS101145	3" LIGHTED WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS102595	4" LIGHTED WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2003
AS101480	3" WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS101146	3" WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS101481	3" LIGHTED WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS102596	4" LIGHTED WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2003
AS103877	4" PLASTIC POLY WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY
AS101147	4" POLY W/ ALUMINUM WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS103907	4" POLY W/ ALUMINUM WHEEL BLACK ANODIZED W/ 5" & 6" FORK 1/2" EXT ASSY
AS101482	4" SOFT ROLL WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS101148	4" SOFT ROLL WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS103878	4" PLASTIC POLY WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY
AS101148	4" POLY W/ ALUMINUM WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS103908	4" POLY W/ ALUMINUM WHEEL BLACK ANODIZED W/ 7" & 8" FORK 1/2" EXT ASSY
AS101149	4" SOFT ROLL WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS103973	4" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY
AS101149	5" POLY W/ ALUMINUM WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS101123	5" POLY W/ BLACK ALUMINUM WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY
AS101150	5" POLY W/ PLASTIC WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS102753	5" SOFT ROLL WHEEL ASSY W/ 5" & 6" FORK 1/2" EXT ASSY
AS103973	5" SOFT ROLL BLACK WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY
AS101151	6" POLY W/ ALUMINUM WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS103938	6" POLY W/ BLACK ALUMINUM WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY
AS101152	6" POLY W/ PLASTIC WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS101153	6" PNEUMATIC W/ ALUMINUM WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS103948	6" PNEUMATIC W/ BLACK ALUMINUM WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY
AS101154	6" PNEUMATIC W/ PLASTIC WHEEL W/ 5" & 6" FORK 1/2" EXT ASSY 2002
AS101802	6" SOFT ROLL WHEEL 1/2" EXT ASSY 2002
AS101155	5" POLY W/ ALUMINUM WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS103924	5" POLY W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY
AS101156	5" POLY W/ PLASTIC WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS102754	5" SOFT ROLL WHEEL ASSY W/ 7" & 8" FORK 1/2" EXT ASSY
AS103974	5" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY
AS101157	6" POLY W/ ALUMINUM WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS103939	6" POLY W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY
AS101158	6" POLY W/ PLASTIC WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS101159	6" PNEUMATIC W/ ALUMINUM WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY

* SEE WHEEL ASSEMBLIES AND FORK ASSEMBLIES FOR INDIVIDUAL COMPONENTS

CASTER COMBINATIONS 2002 (CONT.)

PART NO.	DESCRIPTION
TiLite™ X/SX WITH 1/2" EXTENSION	
AS103949	6" PNEUMATIC W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY
AS101160	6" PNEUMATIC W/ PLASTIC WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS101803	6" SOFT ROLL WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS101161	8" POLY W/ PLASTIC WHEEL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
AS101162	8" PNEU W/ PLASTIC WHL W/ 7" & 8" FORK 1/2" EXT ASSY 2002
TiLite™ X/SX WITH 1" EXTENSION	
AS101163	3" CASTER W/5" AND 6" FORK 1" EXT ASSY
AS101484	3" LIGHTED WHEEL W/5" AND 6" FORK 1" EXT ASSY
AS102597	4" LIGHTED WHEEL W/5" AND 6" FORK 1" EXT ASSY
AS101164	3" CASTER W/7" AND 8" FORK 1" EXT ASSY
AS101485	3" LIGHTED WHEEL W/7" AND 8" FORK 1" EXT ASSY
AS102598	4" LIGHTED WHEEL W/7" AND 8" FORK 1" EXT ASSY
AS103879	4" PLASTIC POLY WHEEL W/ 5" & 6" FORK 1" EXT ASSY
AS101165	4" POLY TIRE W/ALUMINUM CASTER W/5" AND 6" FORK 1" EXT ASSY
AS103909	4" POLY W/ BLACK ALUMINUM WHEEL W/ 5" & 6" FORK 1" EXT ASSY
AS101486	4" SOFT ROLL WHEEL W/5" AND 6" FORK 1" EXT ASSY
AS103959	4" SOFT ROLL WHEEL W/5" AND 6" FORK 1" EXT ASSY
AS103880	4" PLASTIC POLY WHEEL W/ 7" & 8" FORK 1" EXT ASSY
AS101166	4" POLY TIRE W/ALUMINUM CASTER W/7" AND 8" FORK 1" EXT ASSY
AS103910	4" POLY W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK 1" EXT ASSY
AS101487	4" SOFT ROLL WHEEL W/ 7" & 8" FORK 1" EXT ASSY
AS103960	4" SOFT ROLL BLACK WHEEL W/7" AND 8" FORK 1" EXT ASSY
AS101167	5" POLY TIRE W/ALUMINUM CASTER W/5" AND 6" FORK 1" EXT ASSY
AS103925	5" POLY TIRE W/ BLACK ALUMINUM CASTER W/5" AND 6" FORK 1" EXT ASSY
AS101168	5" POLY TIRE W/PLASTIC CASTER W/5" AND 6" FORK 1" EXT ASSY
AS102755	5" SOFT ROLL WHEEL ASSY W/ 5" & 6" FORK 1" EXT ASSY
AS103975	5" SOFT ROLL BLACK WHEEL W/ 5" & 6" FORK 1" EXT ASSY
AS101169	6" POLY TIRE W/ALUMINUM CASTER W/5" AND 6" FORK 1" EXT ASSY
AS101170	6" POLY TIRE W/BLACK ALUMINUM CASTER W/5" AND 6" FORK 1" EXT ASSY
AS101171	6" POLY TIRE W/PLASTIC CASTER W/5" AND 6" FORK 1" EXT ASSY
AS101172	6" PNEU TIRE W/ALUMINUM CASTER W/5" AND 6" FORK 1" EXT ASSY
AS101173	6" PNEU TIRE W/BLACK ALUMINUM CASTER W/5" AND 6" FORK 1" EXT ASSY
AS101174	6" PNEU TIRE W/PLASTIC CASTER W/5" AND 6" FORK 1" EXT ASSY
AS102756	5" SOFT ROLL WHEEL ASSY W/ 7" & 8" FORK 1" EXT ASSY
AS103976	5" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK 1" EXT ASSY
AS101175	6" POLY TIRE W/ALUMINUM CASTER W/7" AND 8" FORK 1" EXT ASSY
AS103941	6" POLY TIRE W/BLACK ALUMINUM CASTER W/7" AND 8" FORK 1" EXT ASSY
AS101176	6" POLY TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1" EXT ASSY
AS101177	6" PNEU TIRE W/ALUMINUM CASTER W/7" AND 8" FORK 1" EXT ASSY
AS103951	6" PNEU TIRE W/BLACK ALUMINUM CASTER W/7" AND 8" FORK 1" EXT ASSY
AS101178	6" PNEU TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1" EXT ASSY
AS101805	6" SOFT ROLL WHEEL W/ 7" & 8" FORK 1" EXT ASSY 2002
AS101179	8" POLY TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1" EXT ASSY
AS101180	8" PNEU TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1" EXT ASSY
TiLite™ X/SX WITH 1-1/2" EXTENSION	
AS101181	3" CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS101488	3" LIGHTED WHEEL W/5" AND 6" FORK 1-1/2" EXT ASSY
AS102599	4" LIGHTED WHEEL W/5" AND 6" FORK 1-1/2" EXT ASSY
AS101182	3" CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101489	3" LIGHTED WHEEL W/7" AND 8" FORK 1-1/2" EXT ASSY
AS102600	4" LIGHTED WHEEL W/7" AND 8" FORK 1-1/2" EXT ASSY
AS103881	4" PLASTIC POLY WHEEL W/ 5" & 6" FORK 1-1/2" EXT ASSY
AS101183	4" POLY TIRE W/ALUM CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS103911	4" POLY TIRE W/BLACK ALUM CASTER W/5" & 6" FORK 1-1/2" EXT ASSY

* SEE WHEEL ASSEMBLIES AND CASTER PIN LOCK FORK ASSEMBLIES
FOR INDIVIDUAL COMPONENTS



CASTER COMBINATIONS 2002 (CONT.)

PART NO.	DESCRIPTION
TiLite™ X/SX WITH 1-1/2" EXTENSION	
AS101490	4" SOFT ROLL WHEEL W/5" AND 6" FORK 1-1/2" EXT ASSY
AS103961	4" SOFT ROLL BLACK WHEEL W/5" AND 6" FORK 1-1/2" EXT ASSY
AS103882	4" PLASTIC POLY WHEEL W/ 7" & 8" FORK 1-1/2" EXT ASSY
AS101184	4" POLY TIRE W/ALUMINUM CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS103912	4" POLY W/ BLACK ALUMINUM WHEEL W/ 7" & 8" FORK 1-1/2" EXT ASSY
AS101491	4" SOFT ROLL WHEEL W/7" AND 8" FORK 1-1/2" EXT ASSY
AS103962	4" SOFT ROLL BLACK WHEEL W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101185	5" POLY TIRE W/ALUMINUM CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS103927	5" POLY TIRE W/BLACK ALUMINUM CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS101186	5" POLY TIRE W/PLASTIC CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS102757	5" SOFT ROLL WHEEL ASSY W/ 5" & 6" FORK 1-1/2" EXT ASSY
AS103977	5" SOFT ROLL BLACK WHEEL W/5" & 6" FORK 1-1/2" EXT ASSY
AS101187	6" POLY TIRE W/ALUMINUM CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS103942	6" POLY TIRE W/BLACK ALUMINUM CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS101188	6" POLY TIRE W/PLASTIC CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS101189	6" PNEU TIRE W/ALUMINUM CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS103952	6" PNEU TIRE W/BLACK ALUMINUM CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS101190	6" PNEU TIRE W/PLASTIC CASTER W/5" AND 6" FORK 1-1/2" EXT ASSY
AS101806	6" SOFT ROLL WHEEL W/ 7" & 8" FORK 1-1/2" EXT ASSY 2002
AS101191	5" POLY TIRE W/ALUMINUM CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS103928	5" POLY TIRE W/BLACK ALUMINUM CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101192	5" POLY TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS102758	5" SOFT ROLL WHEEL ASSY W/ 7" & 8" FORK 1-1/2" EXT ASSY
AS103978	5" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK 1-1/2" EXT ASSY
AS101193	6" POLY TIRE W/ALUMINUM CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS103943	6" POLY TIRE W/BLACK ALUMINUM CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101194	6" POLY TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101195	6" PNEU TIRE W/ALUMINUM CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS103943	6" PNEU TIRE W/BLACK ALUMINUM CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101196	6" PNEU TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101806	6" SOFT ROLL WHEEL W/ 7" & 8" FORK 1-1/2" EXT ASSY 2002
AS101497	8" POLY TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
AS101197	6" PNEU TIRE W/PLASTIC CASTER W/7" AND 8" FORK 1-1/2" EXT ASSY
FROG LEGS ALL TiLite™ CHAIRS	
AS101199	3" CASTER, FROG LEGS
AS101497	3" LIGHTED WHEEL FROG LEGS ASSY 2002
AS102601	4" LIGHTED WHEEL FROG LEGS ASSY 2003
AS103883	4" PLASTIC POLY WHEEL FROG LEGS ASSY
AS101200	4" POLY TIRE W/ALUMINUM CASTER, FROG LEGS
AS103913	4" POLY W/ BLACK ALUMINUM WHEEL FROG LEGS ASSY
AS101493	4" SOFT ROLL WHEEL FROG LEGS ASSY 2002
AS103963	4" SOFT ROLL BLACK WHEEL FROG LEGS ASSY
AS101201	5" POLY TIRE W/ALUMINUM CASTER, FROG LEGS
AS103929	5" POLY TIRE W/ BLACK ALUMINUM CASTER, FROG LEGS
AS101202	5" POLY TIRE W/PLASTIC CASTER, FROG LEGS
AS102759	5" SOFT ROLL WHEEL ASSY W/ FROG LEGS ASSY
AS103979	5" SOFT ROLL BLACK WHEEL W/ FROG LEGS ASSY
FROG LEGS WITH 1/2" EXTENSION ALL TiLite™ CHAIRS	
AS101203	3" WHEEL FROG LEGS W/ 1/2" EXT ASSY 2002
AS101494	3" LIGHTED WHEEL FROG LEGS W/ 1/2" EXT ASSY 2002
AS102602	4" LIGHTED WHEEL FROG LEGS W/ 1/2" EXT ASSY 2003
AS103884	4" PLASTIC POLY WHEEL FROG LEGS W/ 1/2" EXT ASSY
AS101204	4" POLY W/ ALUMINUM WHEEL FROG LEGS W/ 1/2" EXT ASSY 2002
AS103914	4" POLY W/ BLACK ALUMINUM WHEEL FROG LEGS W/ 1/2" EXT ASSY
AS101495	4" SOFT ROLL WHEEL FROG LEGS W/ 1/2" EXT ASSY 2002

* SEE WHEEL ASSEMBLIES AND CASTER PIN LOCK FORK ASSEMBLIES
FOR INDIVIDUAL COMPONENTS



CASTER COMBINATIONS 2002 (CONT.)

PART NO.	DESCRIPTION
FROG LEGS WITH 1/2" EXTENSION ALL TiLite™ CHAIRS	
AS103964	4" SOFT ROLL BLACK WHEEL FROG LEGS W/ 1/2" EXT ASSY
AS101205	5" POLY W/ ALUMINUM WHEEL FROG LEGS W/ 1/2" EXT ASSY 2002
AS103930	5" POLY W/ BLACK ALUMINUM WHEEL FROG LEGS W/ 1/2" EXT ASSY
AS101206	5" POLY W/ PLASTIC WHEEL FROG LEGS W/ 1/2" EXT ASSY 2002
AS102760	5" SOFT ROLL WHEEL ASSY W/ FROG LEGS W/ 1/2" EXT ASSY
AS103980	5" SOFT ROLL BLACK WHEEL W/ FROG LEGS W/ 1/2" EXT ASSY
FROG LEGS WITH 1" EXTENSION ALL TiLite™ CHAIRS	
AS101207	3" WHEEL FROG LEGS W/ 1" EXT ASSY 2002
AS101496	3" LIGHTED WHEEL FROG LEGS W/ 1" EXT ASSY 2002
AS102603	4" LIGHTED WHEEL FROG LEGS W/ 1" EXT ASSY 2003
AS103885	4" PLASTIC POLY WHEEL FROG LEGS W/ 1" EXT ASSY
AS101208	4" POLY W/ ALUMINUM WHEEL FROG LEGS W/ 1" EXT ASSY 2002
AS103915	4" POLY W/ BLACK ALUMINUM WHEEL FROG LEGS W/ 1" EXT ASSY
AS101497	4" SOFT ROLL WHEEL FROG LEGS W/ 1" EXT ASSY 2002
AS103965	4" SOFT ROLL BLACK WHEEL FROG LEGS W/ 1" EXT ASSY
AS101209	5" POLY W/ ALUMINUM WHEEL FROG LEGS W/ 1" EXT ASSY 2002
AS103931	5" POLY W/ BLACK ALUMINUM WHEEL FROG LEGS W/ 1" EXT ASSY
AS101210	5" POLY W/ PLASTIC WHEEL FROG LEGS W/ 1" EXT ASSY 2002
AS102761	5" SOFT ROLL WHEEL ASSY W/ FROG LEGS W/ 1" EXT ASSY
AS103981	5" SOFT ROLL BLACK WHEEL FROG LEGS W/ 1" EXT ASSY
FROG LEGS WITH 1-1/2" EXTENSION ALL TiLite™ CHAIRS	
AS101211	3" WHEEL FROG LEGS W/ 1-1/2" EXT ASSY 2002
AS101498	3" LIGHTED WHEEL FROG LEGS W/ 1-1/2" EXT ASSY 2002
AS102604	4" LIGHTED WHEEL FROG LEGS W/ 1-1/2" EXT ASSY 2003
AS103886	4" PLASTIC POLY WHEEL FROG LEGS W/ 1-1/2" EXT ASSY
AS101212	4" POLY W/ ALUMINUM WHEEL FROG LEGS W/ 1-1/2" EXT ASSY 2002
AS103916	4" POLY W/ BLACK ALUMINUM WHEEL FROG LEGS W/ 1-1/2" EXT ASSY
AS101499	4" SOFT ROLL WHEEL FROG LEGS W/ 1-1/2" EXT ASSY 2002
AS103966	4" SOFT ROLL BLACK WHEEL FROG LEGS W/ 1-1/2" EXT ASSY
AS101213	5" POLY W/ ALUMINUM WHEEL FROG LEGS W/ 1-1/2" EXT ASSY 2002
AS103932	5" POLY W/ BLACK ALUMINUM WHEEL FROG LEGS W/ 1-1/2" EXT ASSY
AS101214	5" POLY W/ PLASTIC WHEEL FROG LEGS W/ 1-1/2" EXT ASSY 2002
AS102762	5" SOFT ROLL WHEEL ASSY W/ FROG LEGS W/ 1-1/2" EXT ASSY
AS103982	5" SOFT ROLL BLACK WHEEL W/ FROG LEGS W/ 1-1/2" EXT ASSY
CASTER PIN LOCKS, TiLite™ TR/TRC/XC	
AS101215	3" WHEEL W/ CASTER PIN LOCK ASSY 2002
AS101500	3" LIGHTED WHEEL W/ CASTER PIN LOCK ASSY 2002
AS103887	4" PLASTIC POLY WHEEL W/ CASTER PIN LOCK ASSY
AS101216	4" WHEEL W/ CASTER PIN LOCK ASSY 2002
AS103917	4" BLACK WHEEL W/ CASTER PIN LOCK ASSY
AS101501	4" SOFT ROLL WHEEL W/ CASTER PIN LOCK ASSY 2002
AS103967	4" SOFT ROLL BLACK WHEEL W/ CASTER PIN LOCK ASSY
AS101217	5" POLY W/ ALUM WHEEL W/ SPEC. FORK CASTER PIN LOCK ASSY 2002
AS103933	5" POLY W/ BLACK ALUM WHEEL W/ SPEC. FORK CASTER PIN LOCK ASSY
AS101218	5" POLY W/ PLASTIC WHEEL W/ SPEC. FORK CASTER PIN LOCK ASSY 2002
AS102763	5" SOFT ROLL WHEEL ASSY W/ SPEC. FORK CASTER PIN LOCK ASSY
AS103983	5" SOFT ROLL BLACK WHEEL W/ SPEC. FORK CASTER PIN LOCK ASSY
CASTER PIN LOCKS, TiLite™ X/SX/XC	
AS101219	5" POLY W/ ALUM WHEEL W/ CASTER PIN LOCK ASSY 2002
AS103934	5" POLY W/ BLACK ALUM WHEEL W/ CASTER PIN LOCK ASSY
AS101220	5" POLY W/ PLASTIC WHEEL W/ CASTER PIN LOCK ASSY 2002
AS102764	5" SOFT ROLL WHEEL ASSY W/ CASTER PIN LOCK ASSY
AS103984	5" SOFT ROLL BLACK WHEEL W/ CASTER PIN LOCK ASSY

* SEE WHEEL ASSEMBLIES AND CASTER PIN LOCK FORK ASSEMBLIES FOR INDIVIDUAL COMPONENTS



Parts Manual

CASTER COMBINATIONS 2002 (CONT.)

PART NO.	DESCRIPTION
CASTER PIN LOCKS, TiLite™ X/SX/XC	
AS101221	6" POLY W/ ALUM WHEEL W/ CASTER PIN LOCK ASSY 2002
AS103944	6" POLY W/ BLACK ALUM WHEEL W/ CASTER PIN LOCK ASSY 2002
AS101222	6" POLY W/ PLASTIC WHEEL W/ CASTER PIN LOCK ASSY 2002
AS101223	6" PNEU W/ ALUM WHEEL W/ CASTER PIN LOCK ASSY 2002
AS103954	6" PNEU W/ BLACK ALUM WHEEL W/ CASTER PIN LOCK ASSY 2002
AS101224	6" PNEU W/ PLASTIC WHEEL W/ CASTER PIN LOCK ASSY 2002
AS101808	6" SOFT ROLL WHEEL W/ CASTER PIN LOCK ASSY 2002
CASTER PIN LOCKS, TiLite™ X/SX	
AS101225	3" WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2002
AS101502	3" LIGHTED WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2002
AS102606	4" LIGHTED WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2003
AS101226	3" WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101503	3" LIGHTED WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS102607	4" LIGHTED WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2003
AS103888	4" PLASTIC POLY WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2002
AS101227	4" WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2002
AS103918	4" BLACK WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2002
AS101504	4" SOFT ROLL WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2002
AS103968	4" SOFT ROLL BLACK WHEEL W/ 5" & 6" FORK CASTER PIN LOCK ASSY 2002
AS103889	4" PLASTIC POLY WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101228	4" WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS103919	4" BLACK WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101505	4" SOFT ROLL WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS103969	4" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101229	5" POLY W/ ALUM WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS103935	5" POLY W/ BLACK ALUM WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101230	5" POLY W/ PLASTIC WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101235	5" SOFT ROLL WHEEL ASSY W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS103935	5" SOFT ROLL BLACK WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101231	6" POLY W/ ALUM WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002
AS103945	6" POLY W/ BLACK ALUM WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002
AS101232	6" POLY W/ PLASTIC WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002
AS101233	6" PNEU W/ ALUM WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002
AS103955	6" PNEU W/ BLACK ALUM WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002
AS101234	6" PNEU W/ PLASTIC WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002
AS101239	6" SOFT ROLL WHEEL W/ 7" & 8" FORK CASTER PIN LOCK ASSY 2002
AS101235	8" POLY W/ PLASTIC WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002
AS101236	8" PNEU W/ PLASTIC WHEEL W/ 7" & 8" CASTER PIN LOCK ASSY 2002